


1

Attach electrodes to each electrode connector




2

Remove plastic from electrodes

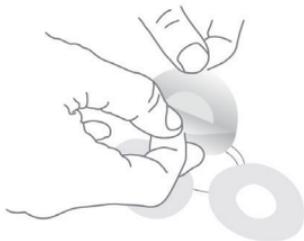

3

Attach device to patient

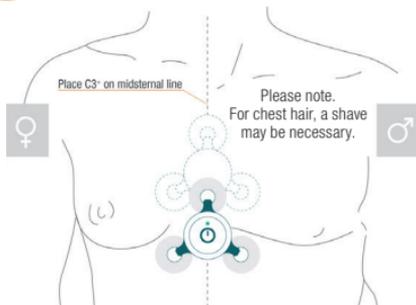

Make sure the adhesive sticks properly to skin.

4

Start recording



A

Button presses




Single press
Turn on / Mark event

Double press
Activate Bluetooth

Long press (3 seconds)
Turn off (Note: Device must not have connection to body)

B

LED light notifications

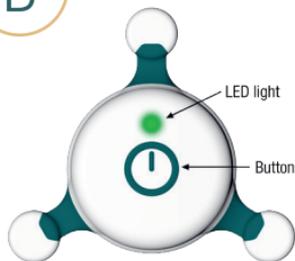

Green Light
Slow: Recording in progress
Fast: Turning ON / Event button pressed
Constant: Fully charged

Blue Light
Slow: Streaming Bluetooth
Fast: Pairing with Bluetooth

Yellow Light
Slow: Memory full
Fast: Low battery / Turning off
Constant: Charging


White Light
Fast: Lead OFF detected, C3+ does not have proper contact to body.
Constant: Error. Contact Cortrium or supplier

C


Included components



C3+ recording devices

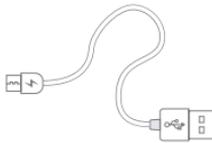

USB-Cable

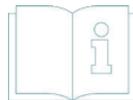

Instructions

Support: www.cortrium.com | support@cortrium.com

Cortrium C3+ Holter Monitor | SHORT MANUAL | Version: 1.0 UK - Date: 2019-10-14