Cortrium Apex 1.22

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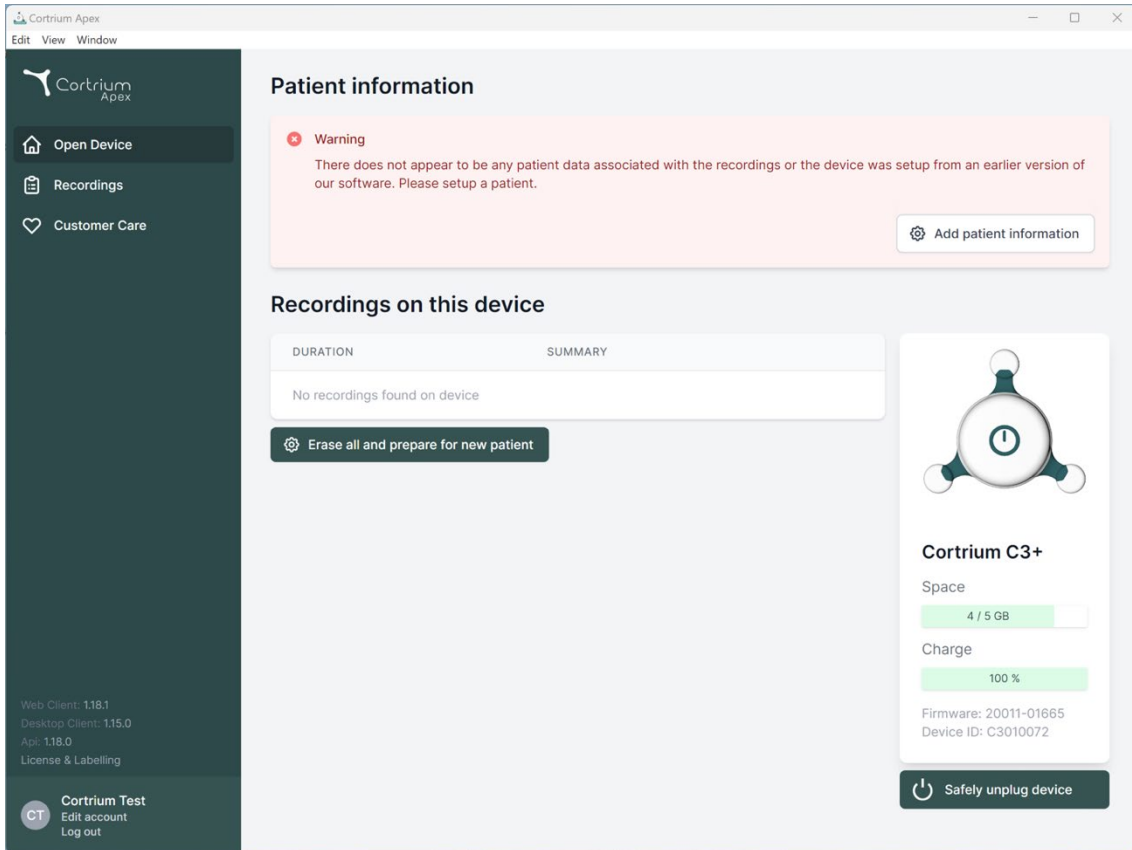
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' displays 'Space' (4 / 5 GB) and 'Charge' (100 %) with progress bars, and includes 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning This will erase all recordings on the device

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
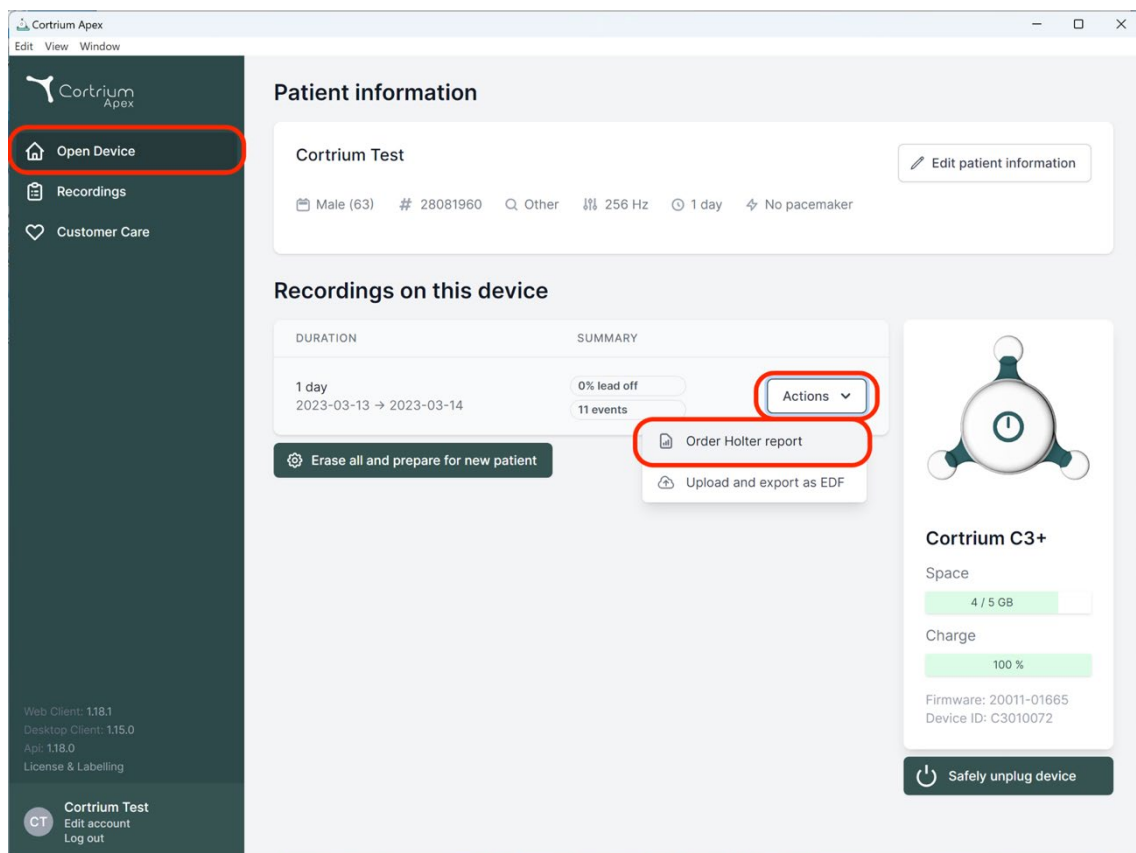
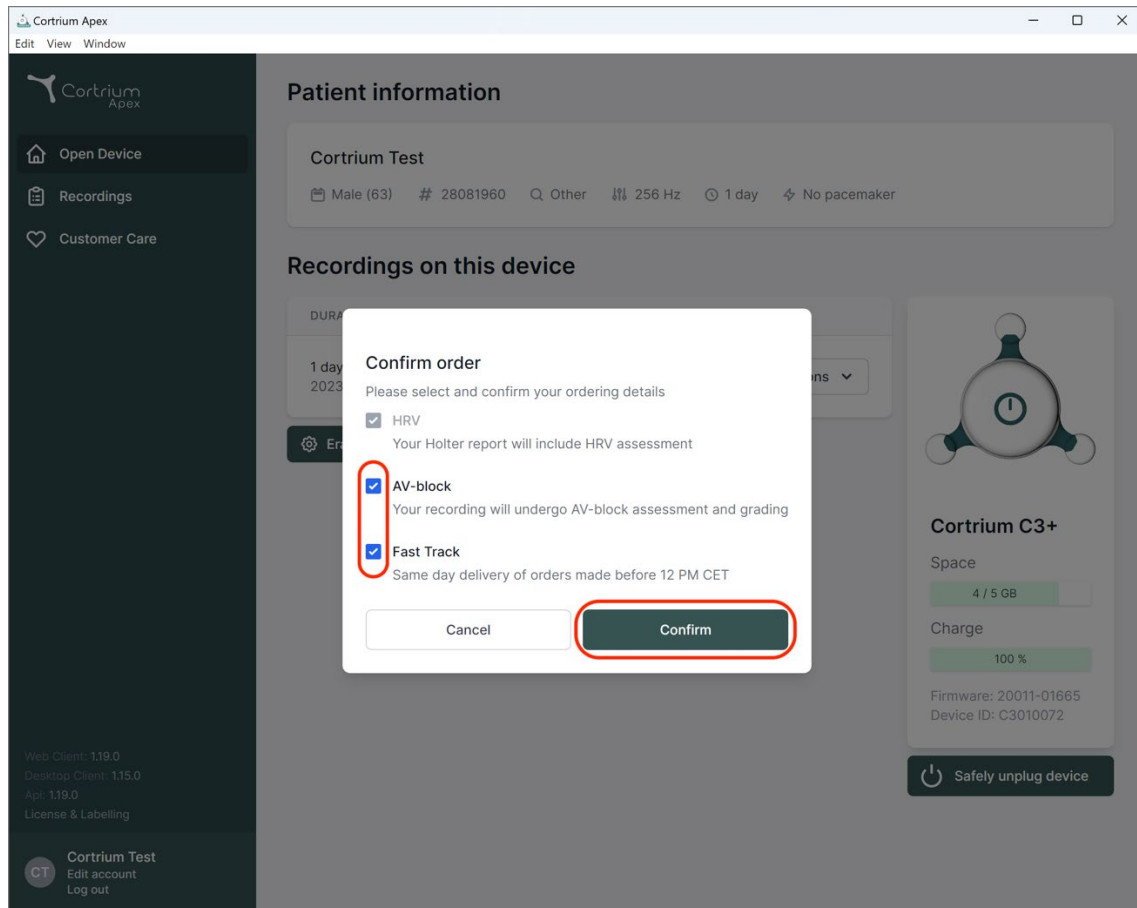
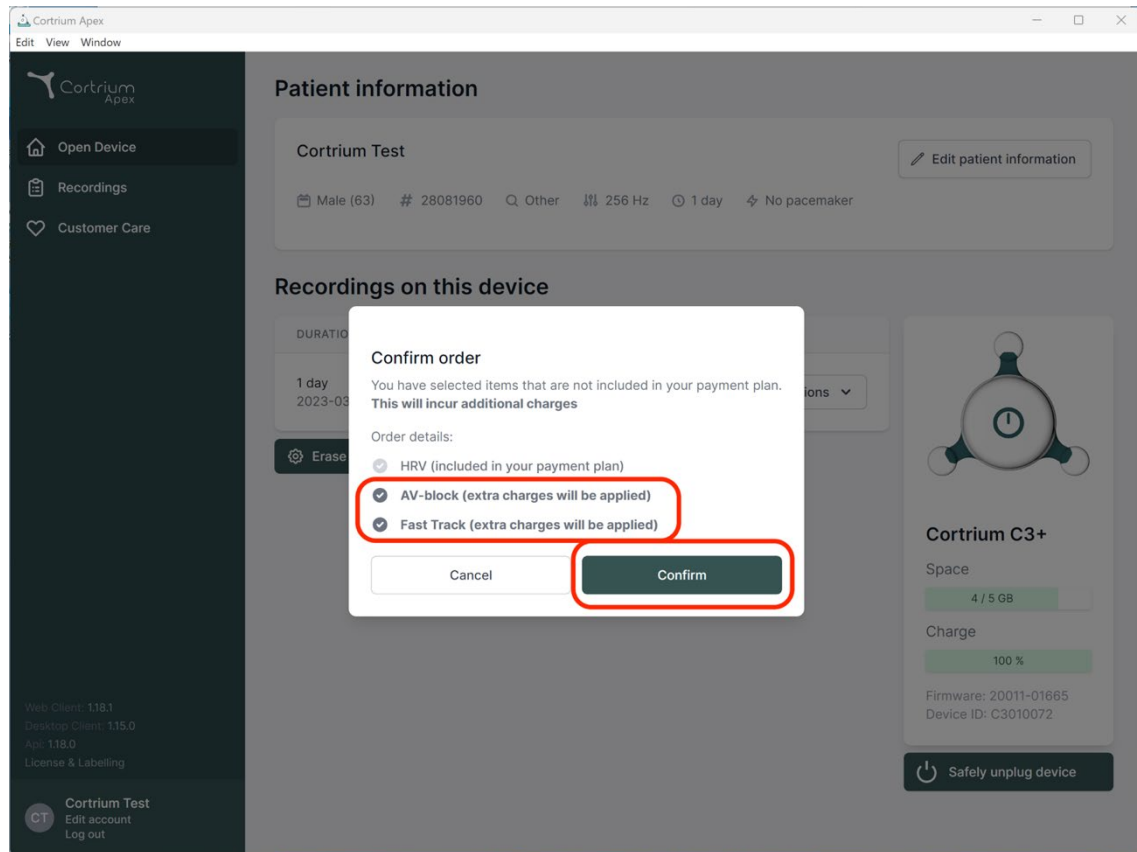
Herefter uploades optagelsen.

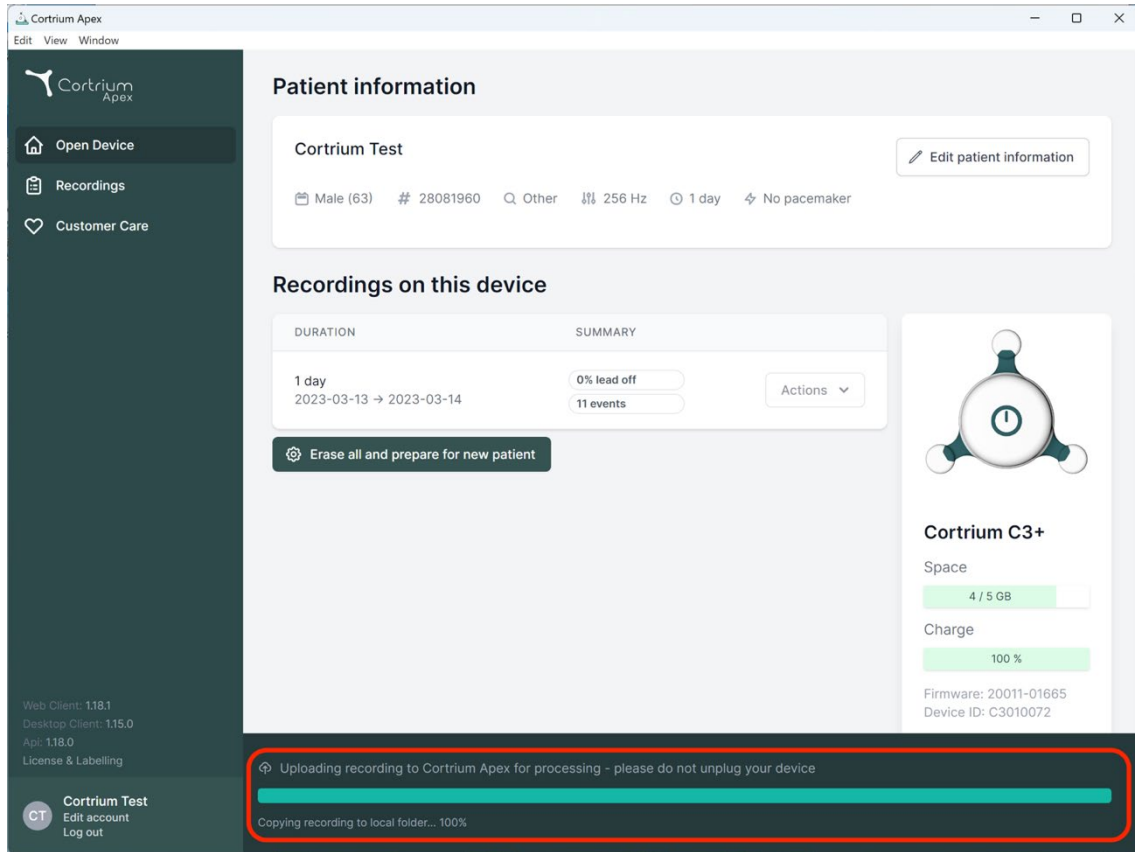The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are system status indicators: 'Web Client: 118.1', 'Desktop Client: 115.0', 'Api: 118.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with an 'Edit account' and 'Log out' option. The main content area is titled 'Patient information' and displays data for 'Cortrium Test', including 'Male (63)', ID '28081960', 'Other' category, '256 Hz' sampling rate, '1 day' duration, and 'No pacemaker'. An 'Edit patient information' button is present. Below this is the 'Recordings on this device' section, which shows a table with columns for 'DURATION' and 'SUMMARY'. A recording for '1 day' (2023-03-13 to 2023-03-14) is listed with '0% lead off' and '11 events'. An 'Actions' dropdown menu is next to it. A button 'Erase all and prepare for new patient' is also visible. To the right of the table is a device status card for 'Cortrium C3+' showing 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. At the bottom of the interface, a red-bordered box highlights a status bar with the text 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar. Below this, it says 'Copying recording to local folder... 100%'.

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
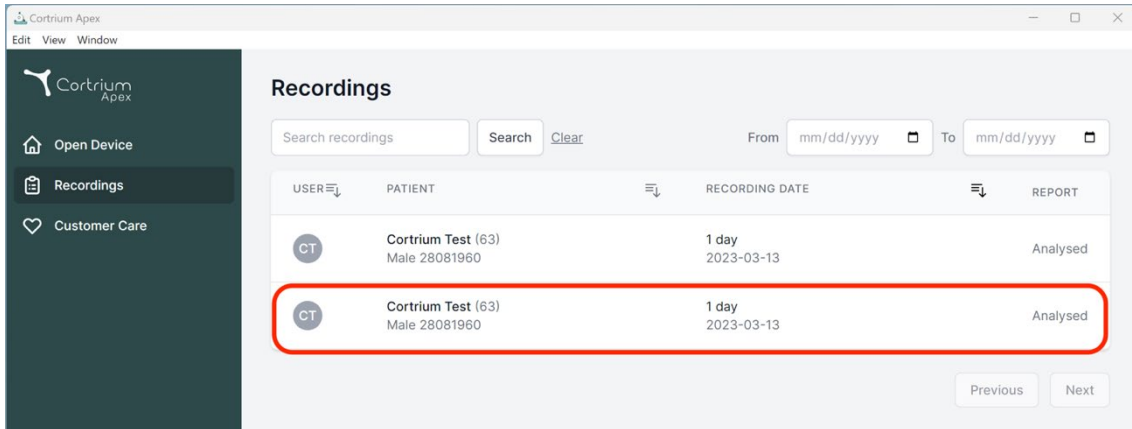
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

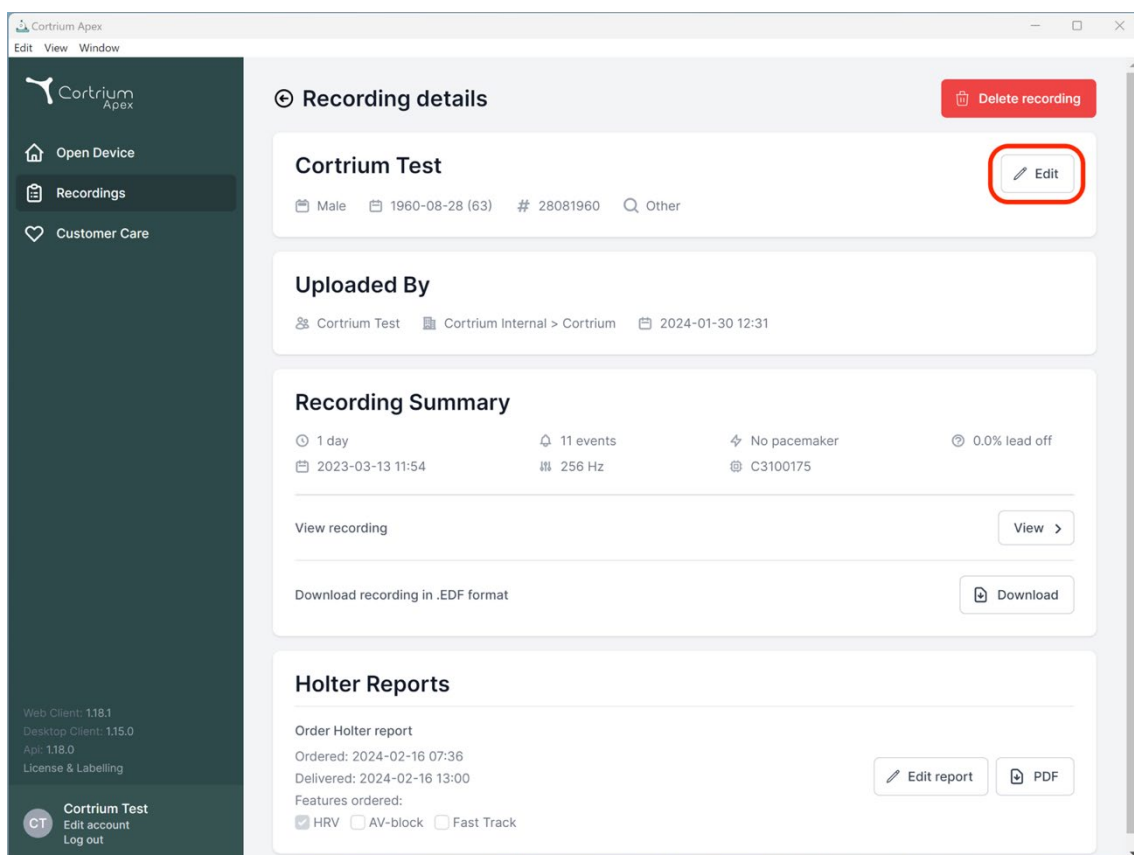Patientoplysningerne kan redigeres ved at klikke på knappen "Rediger".

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary
 1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports
 Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
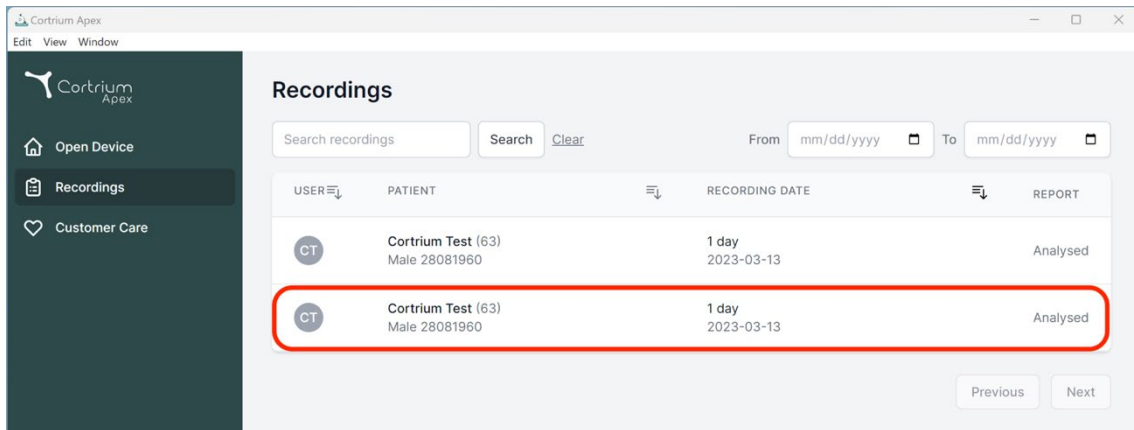
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

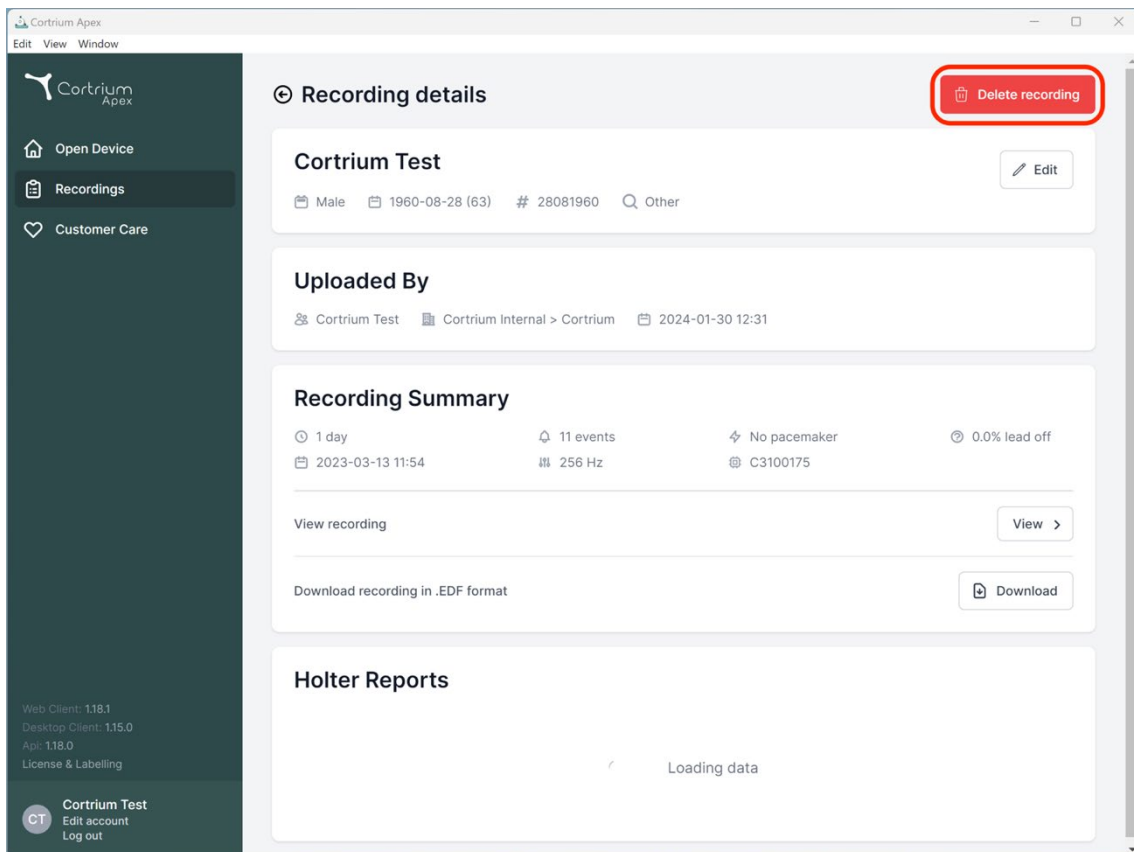Recording details

Cortrium Test [Edit]

Male 1960-08-28 (63) # 28081960 [Other]

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View >]

Download recording in .EDF format [Download]

Holter Reports

Loading data

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

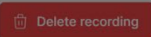
 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

Cortrium Test

 Male  1960-08-28 (63)  # 28081960  Other 

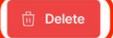
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete


You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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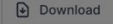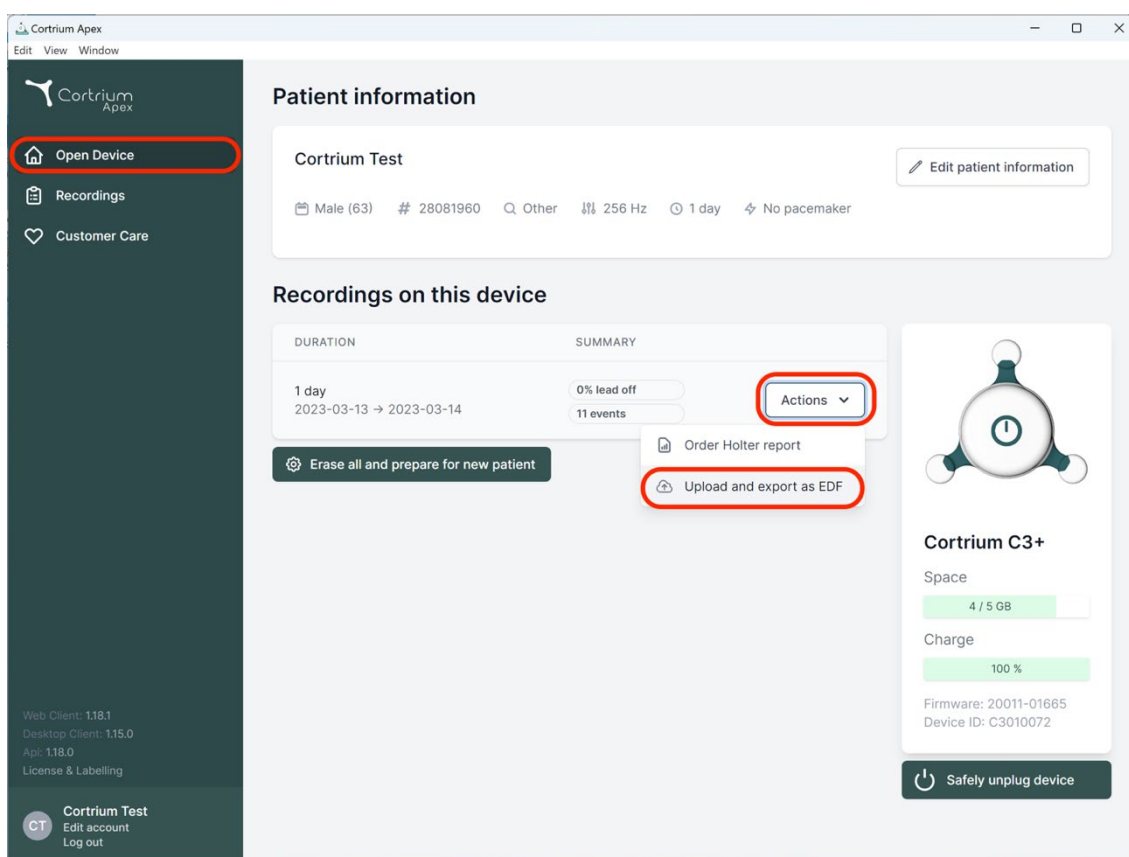
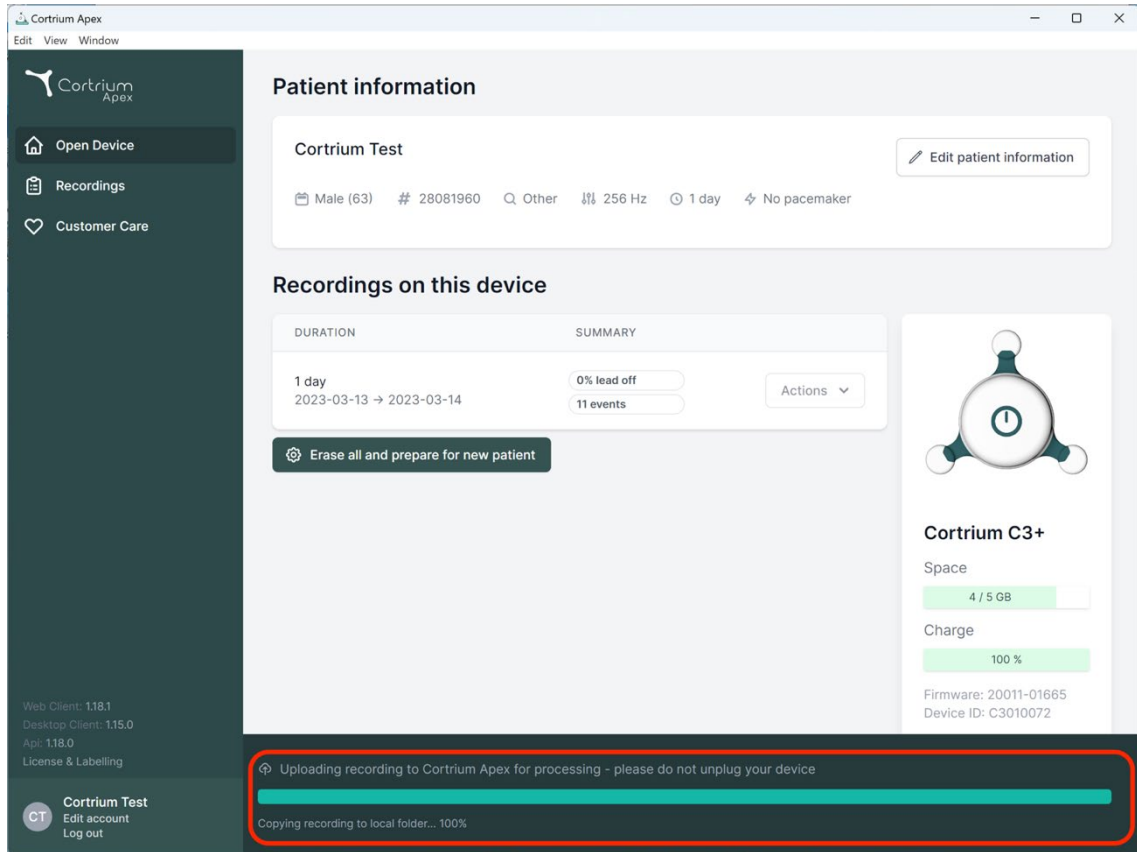
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

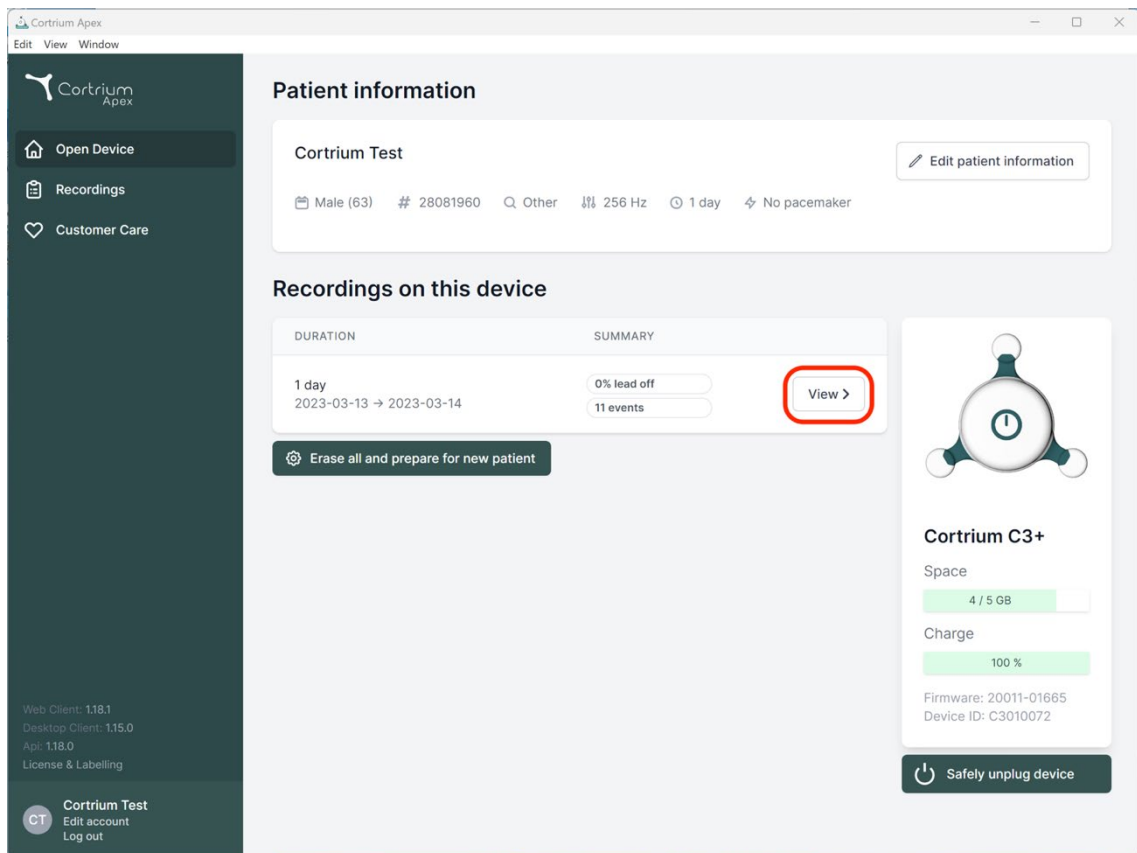
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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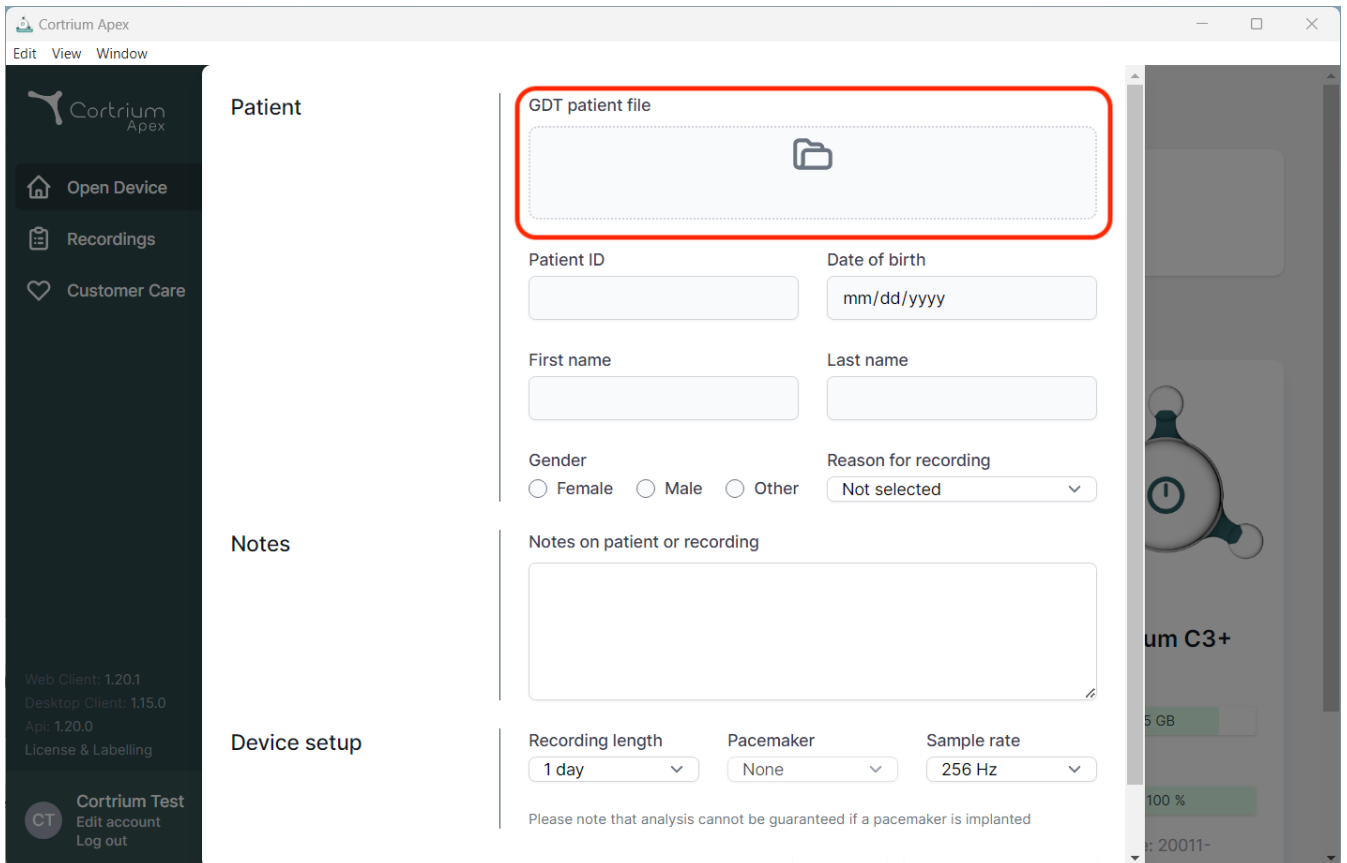
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted and contains a form for entering patient information. The form includes a red-bordered box for the GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. Below these are radio buttons for Gender (Female, Male, Other) and a dropdown for Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdowns for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

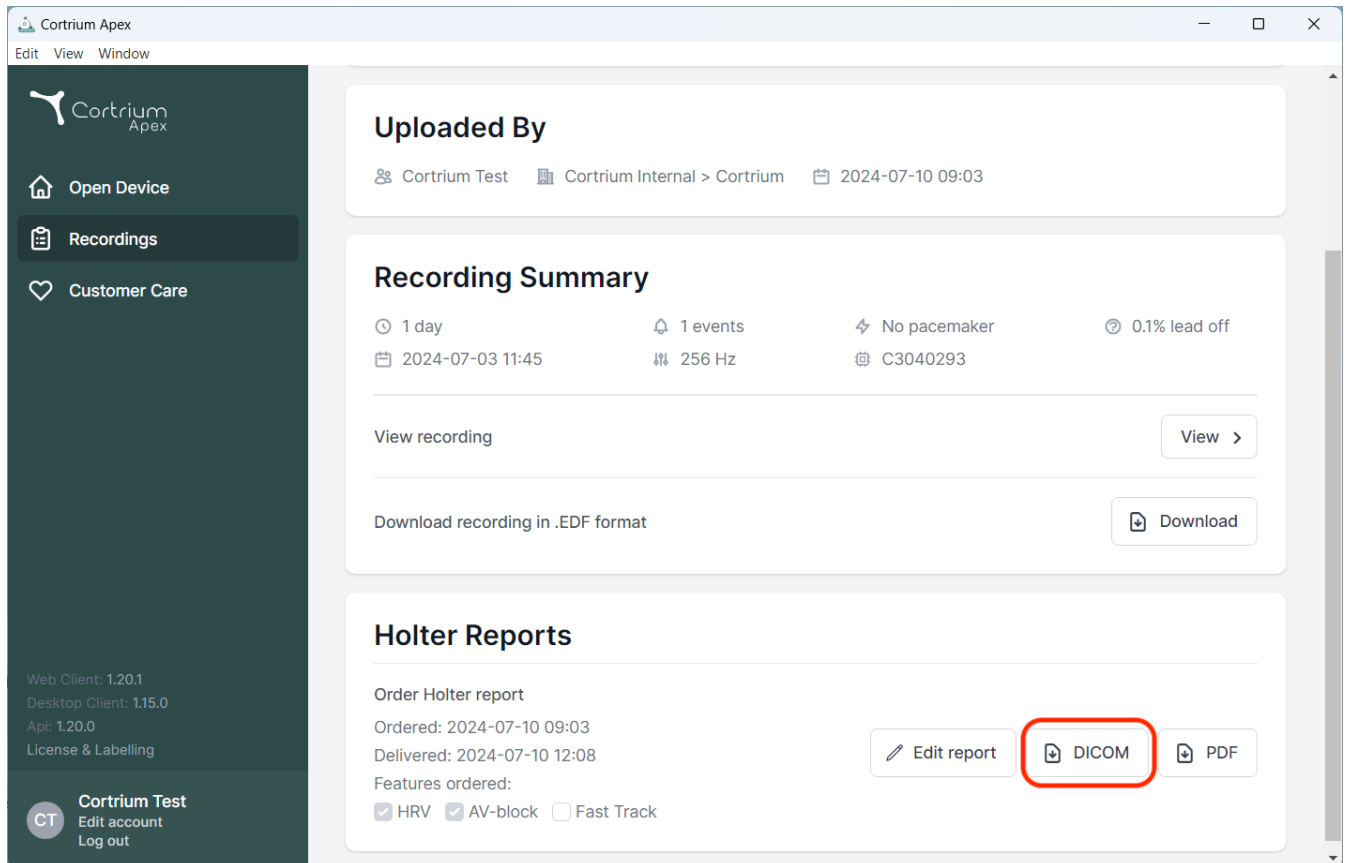
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying recording details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (which is highlighted with a red rectangle), and 'PDF'. Below these buttons, the 'Features ordered' section shows checkboxes for 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked). The bottom of the sidebar contains version information and a user profile for 'Cortrium Test'.

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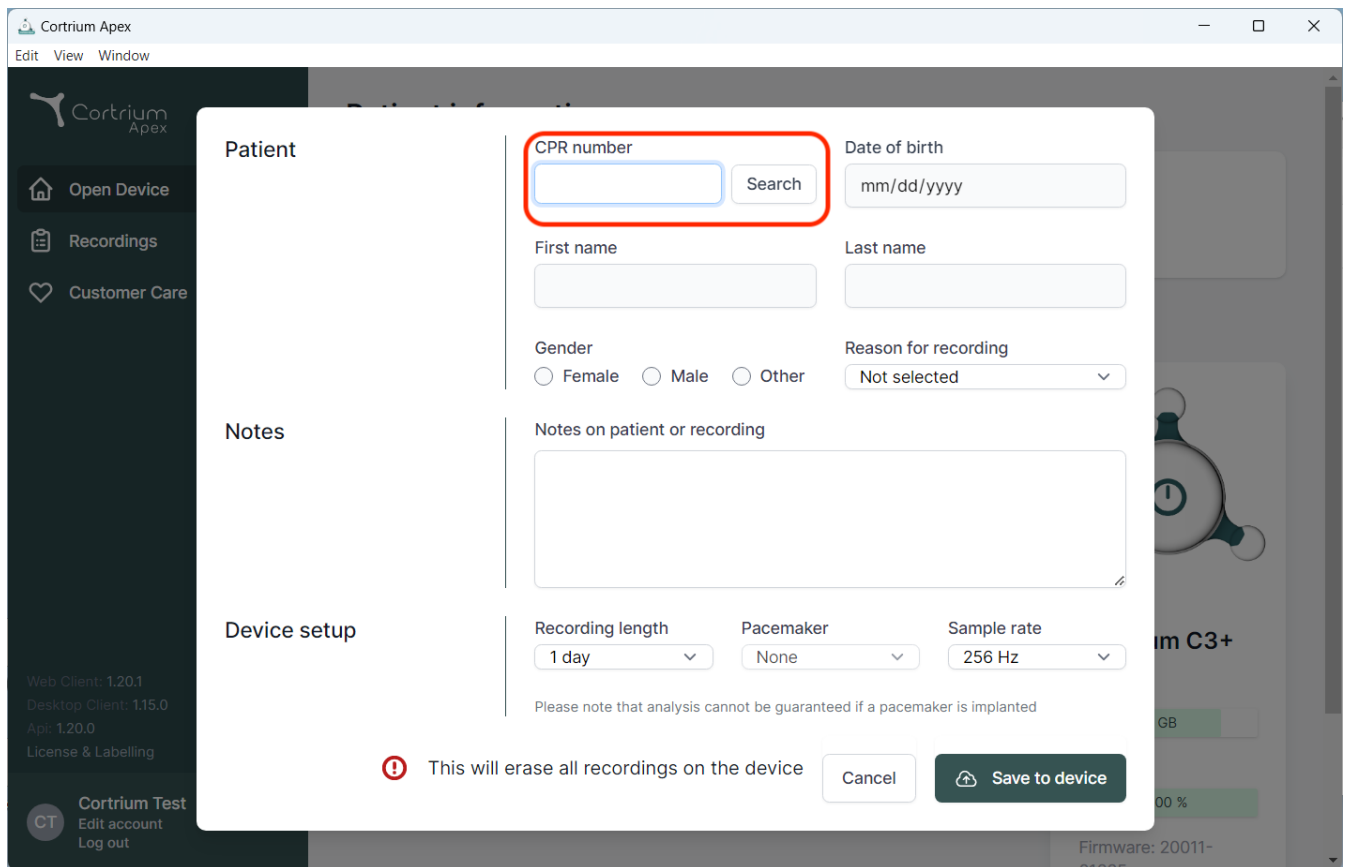The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for patient information. The form is divided into sections: "Patient", "Notes", and "Device setup".

Patient section:

- CPR number:** A text input field with a red border and a "Search" button next to it.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes section:

- Notes on patient or recording:** A large text area for notes.

Device setup section:

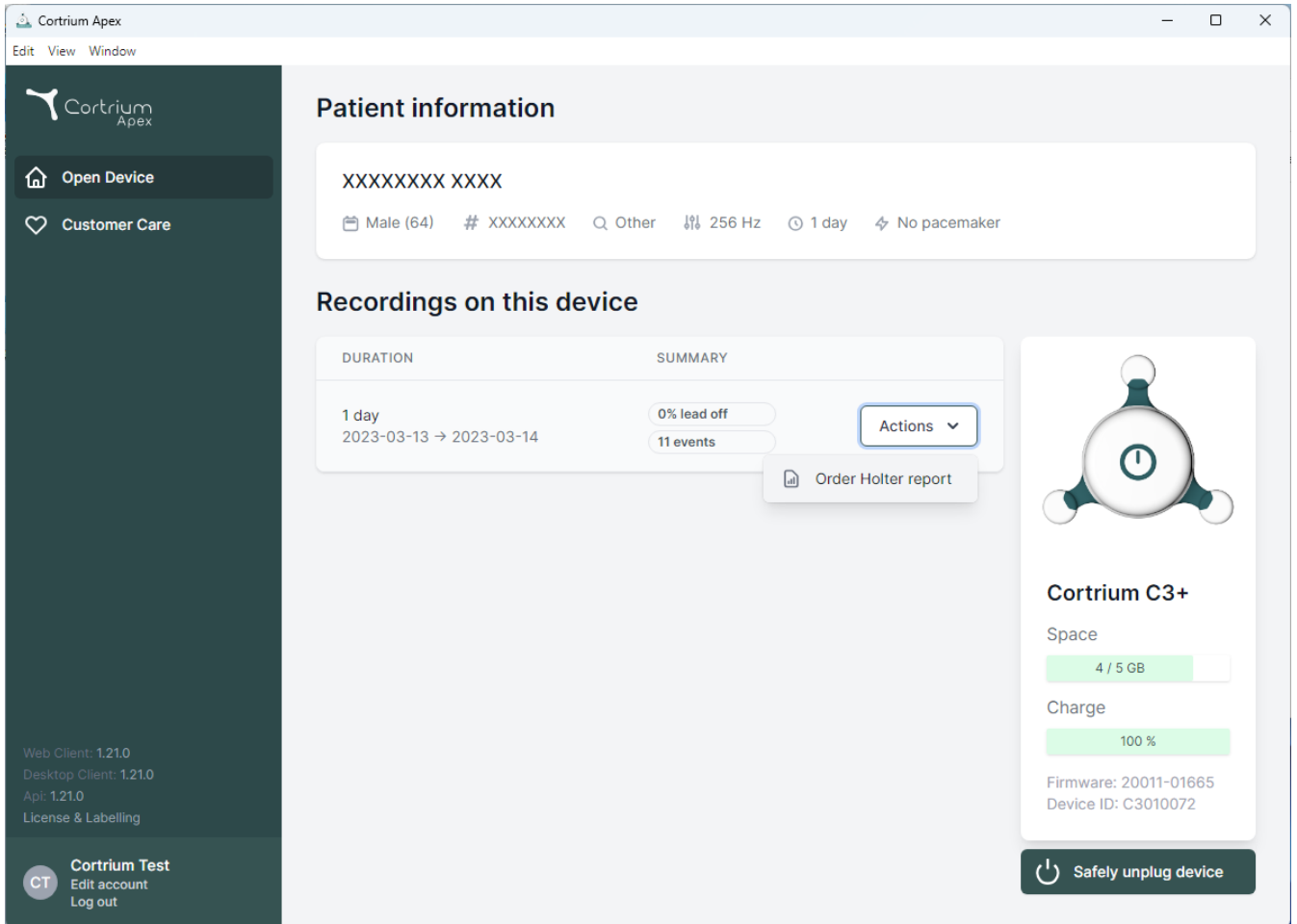
- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the modal, there is a red warning icon and the text: "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions

[Order Holter report](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

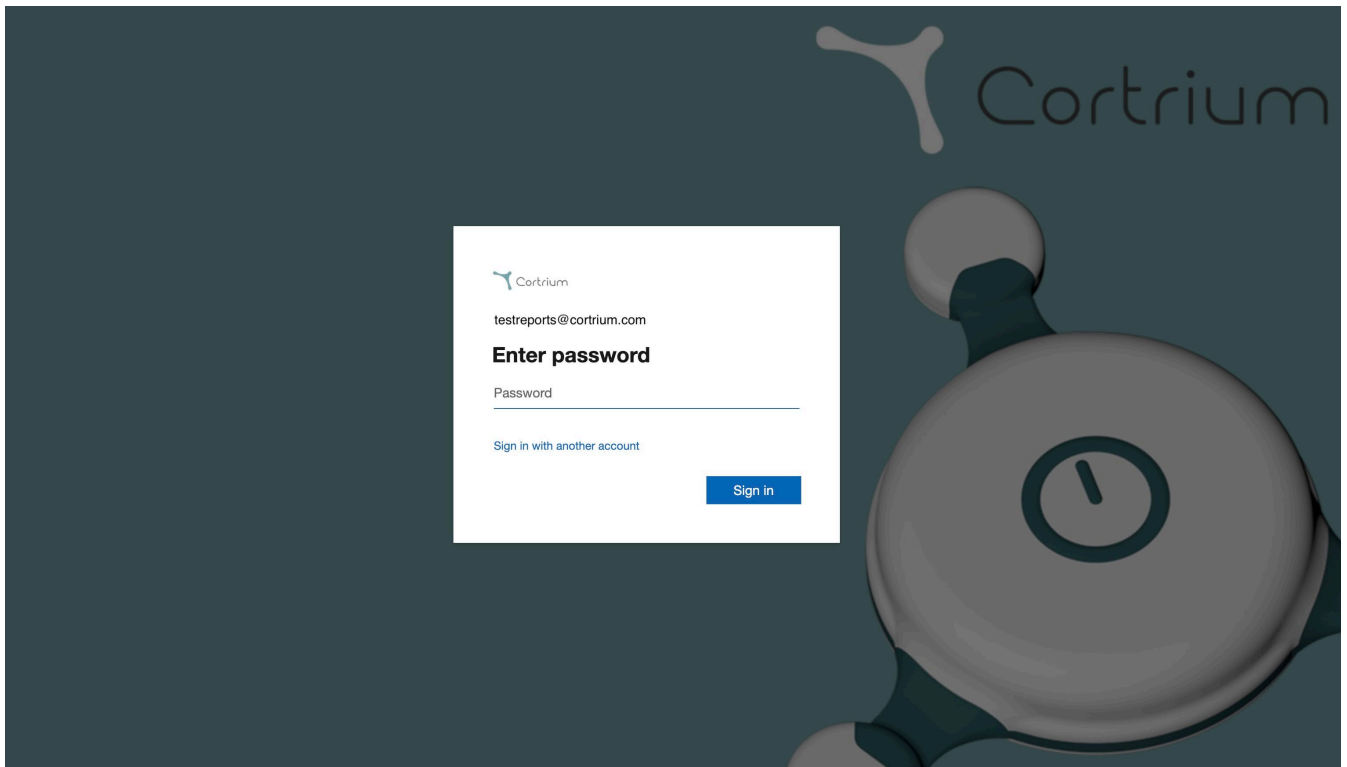
[Safely unplug device](#)

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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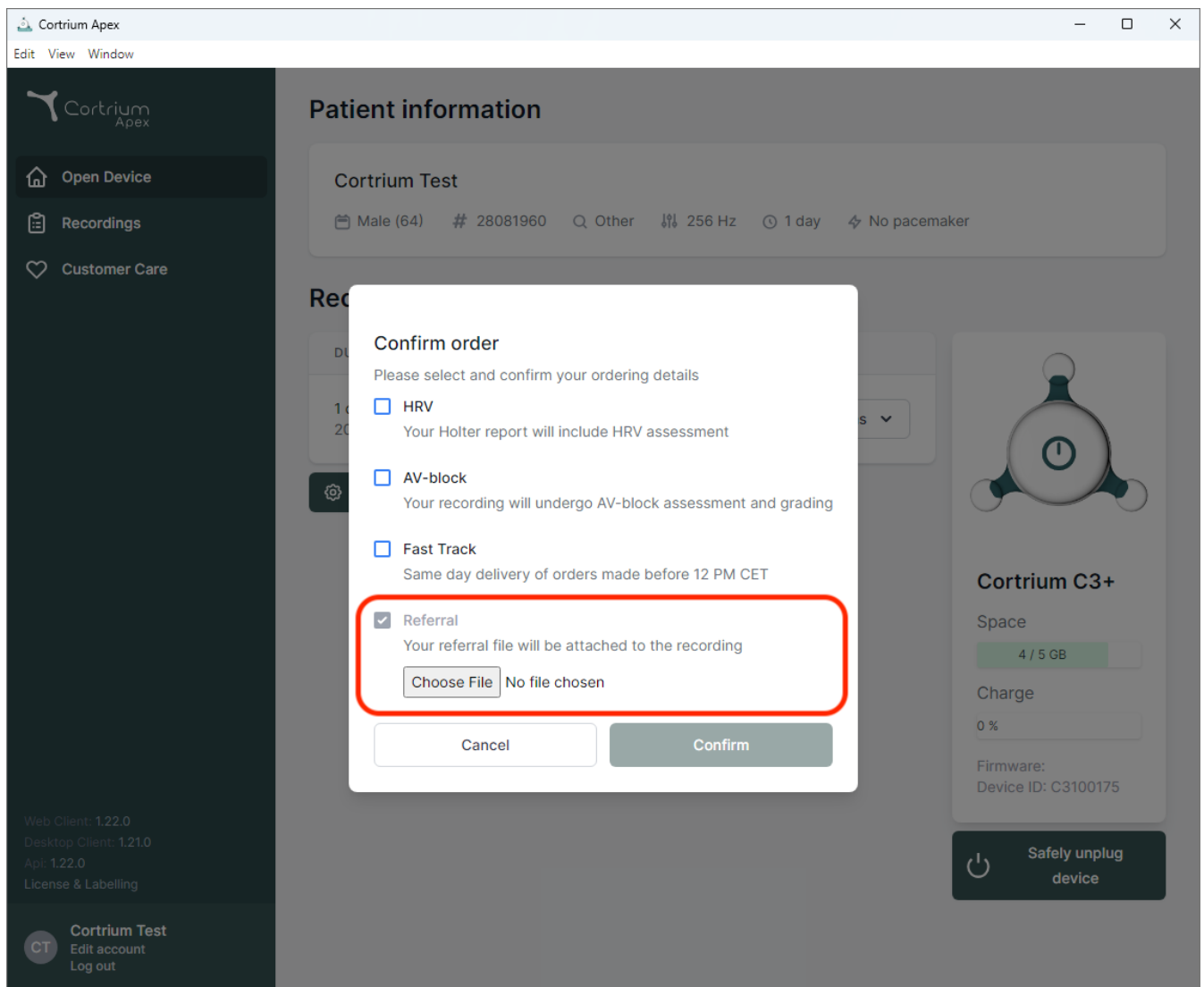
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
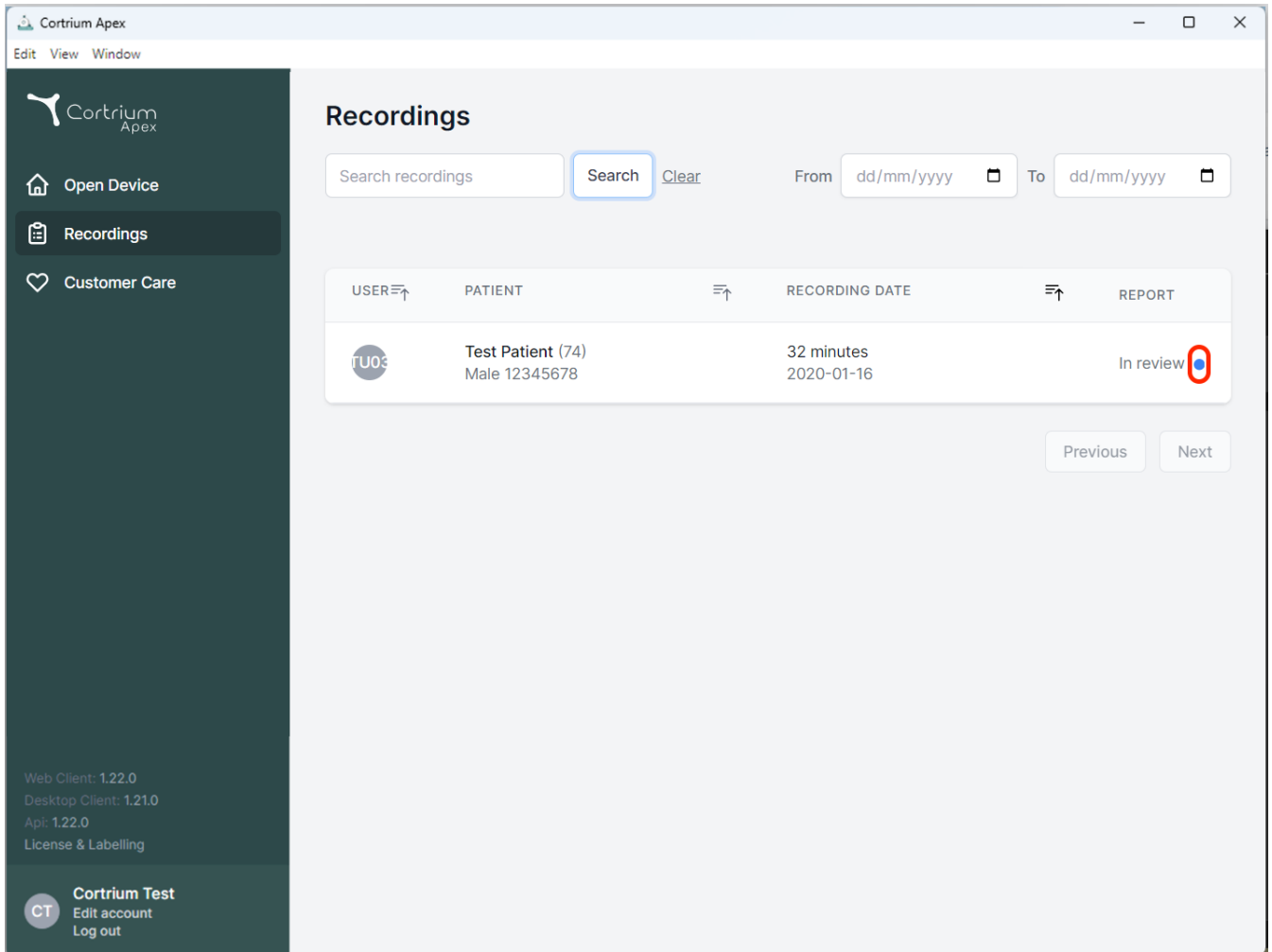
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (selected), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot is visible next to the "In review" status. At the bottom right of the table are "Previous" and "Next" buttons. The bottom of the sidebar shows version information and a user profile for "Cortrium Test".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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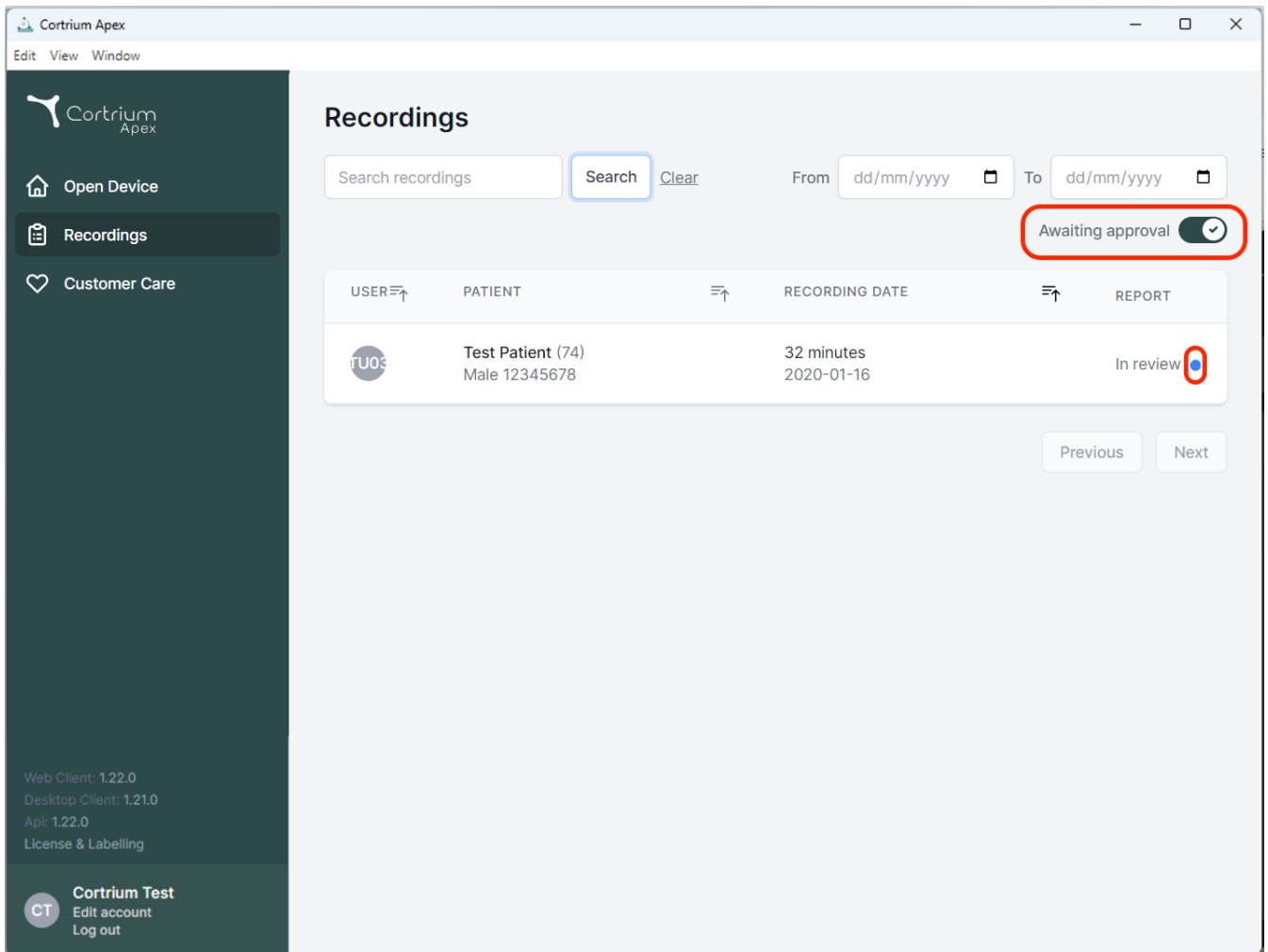Search recordings [Clear](#) From To

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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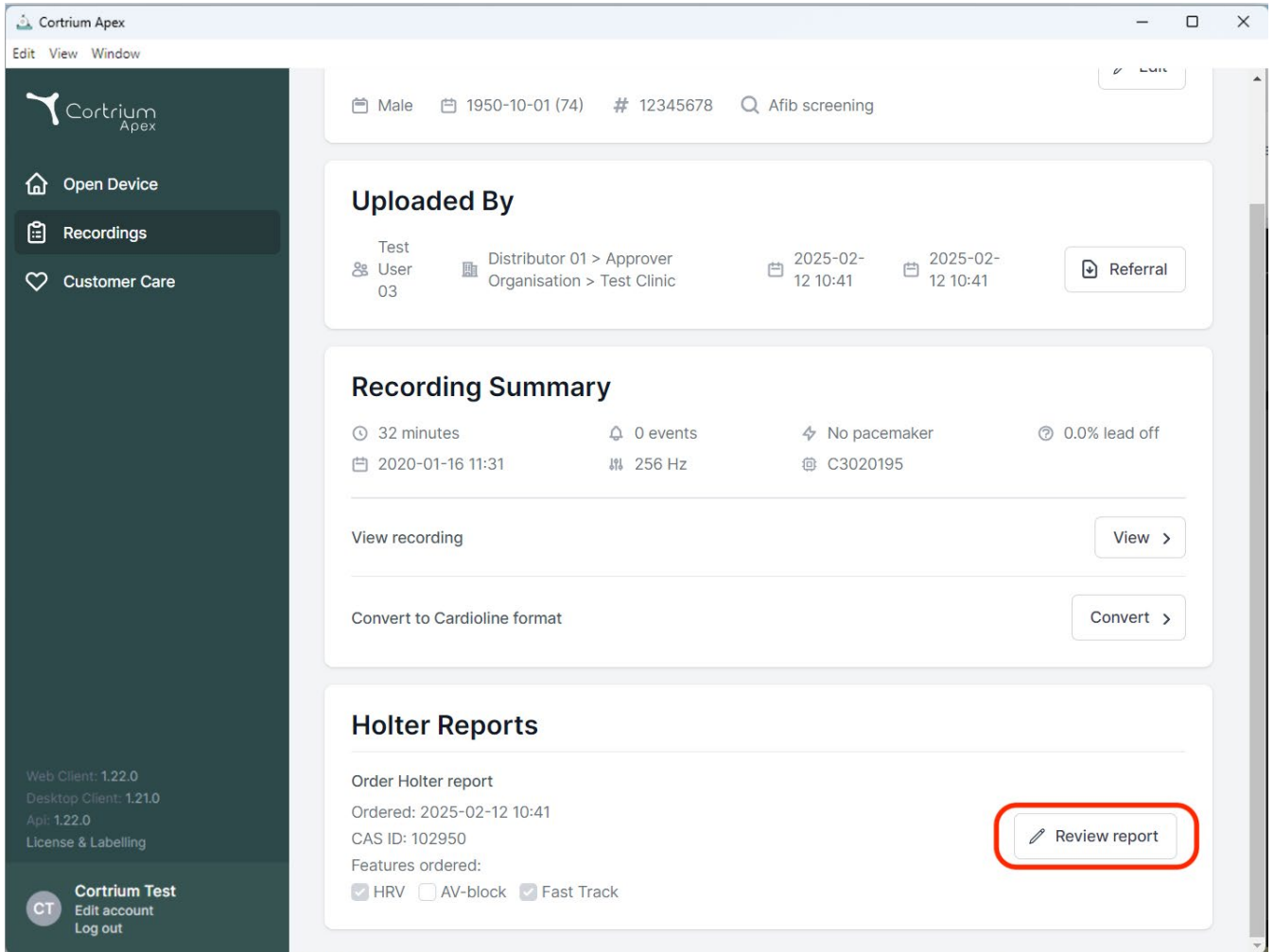
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the "Uploaded By" section, showing the test user (User 03) and the distributor (Distributor 01 > Approver Organisation > Test Clinic), with upload dates of 2025-02-12 10:41 and a "Referral" button. The "Recording Summary" section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for "View recording" and "Convert to Cardioline format". The "Holter Reports" section shows the order details: "Order Holter report", "Ordered: 2025-02-12 10:41", "CAS ID: 102950", and "Features ordered: HRV, AV-block, Fast Track". A "Review report" button is highlighted with a red rectangle.

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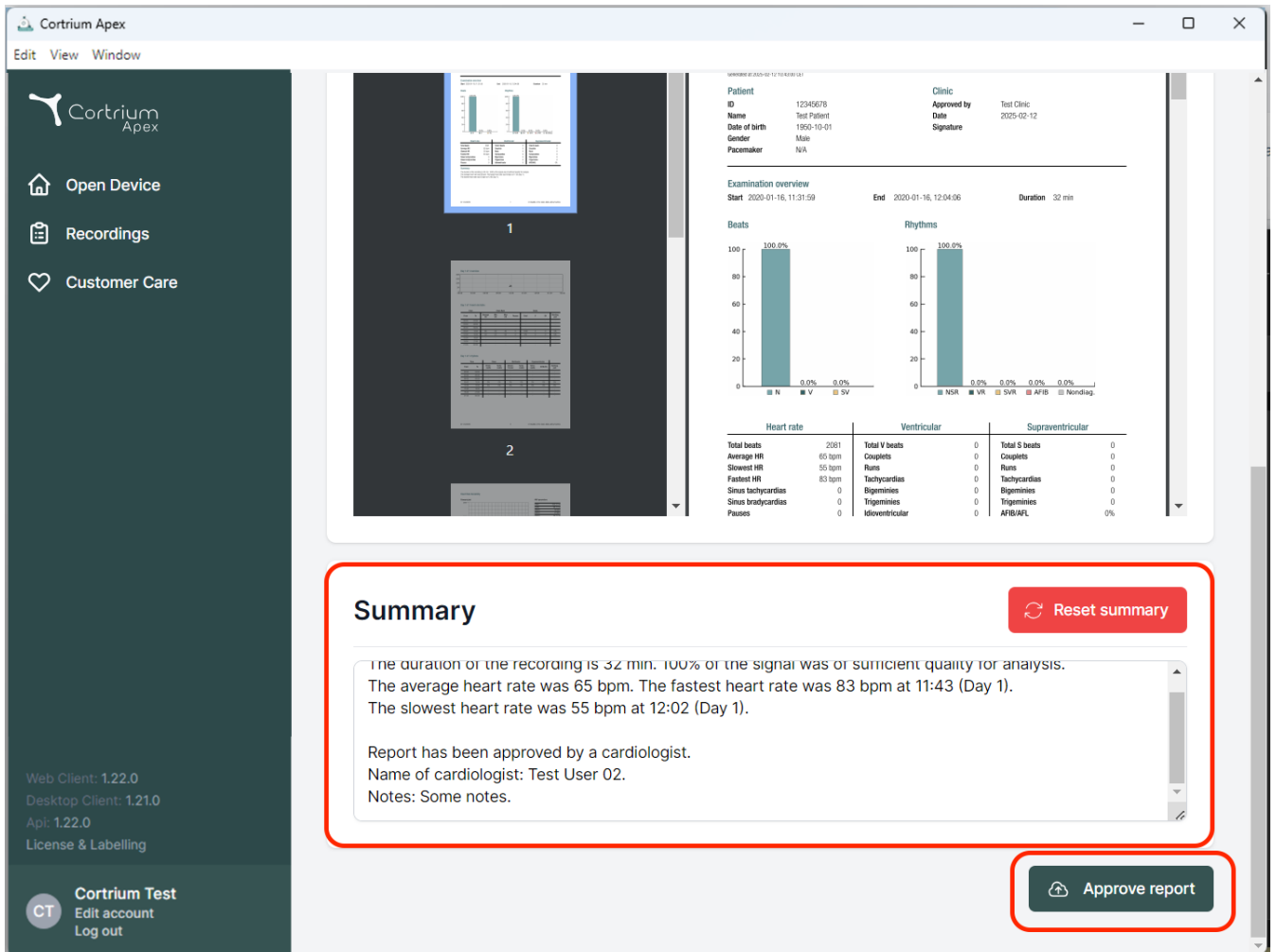Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

Beats
100.0%
0.0% 0.0% 0.0%

Rhythms
100.0%
0.0% 0.0% 0.0% 0.0% 0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AFIB/AFI	0%

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
E-mail: info@cortrium.com • Hjemmeside: www.cortrium.com

Cortrium Apex UDI(01)05745000379033