Cortrium Apex 1.22

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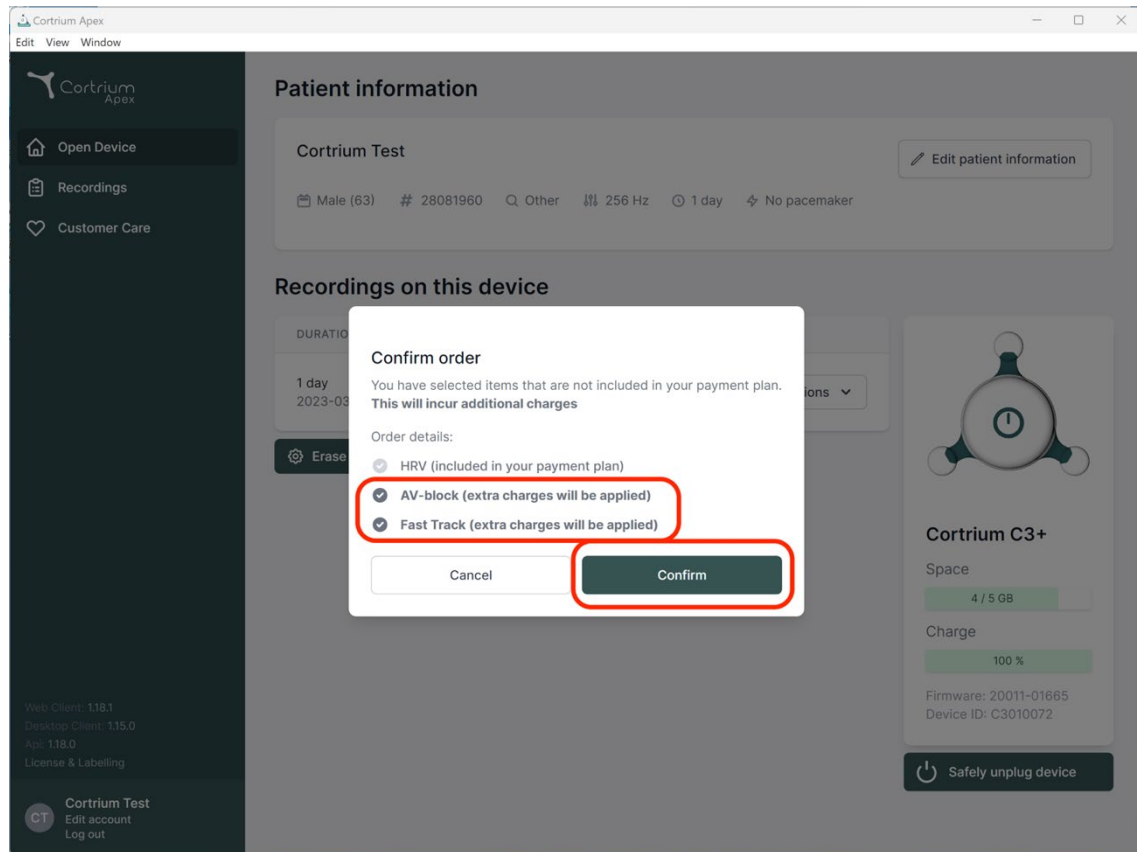
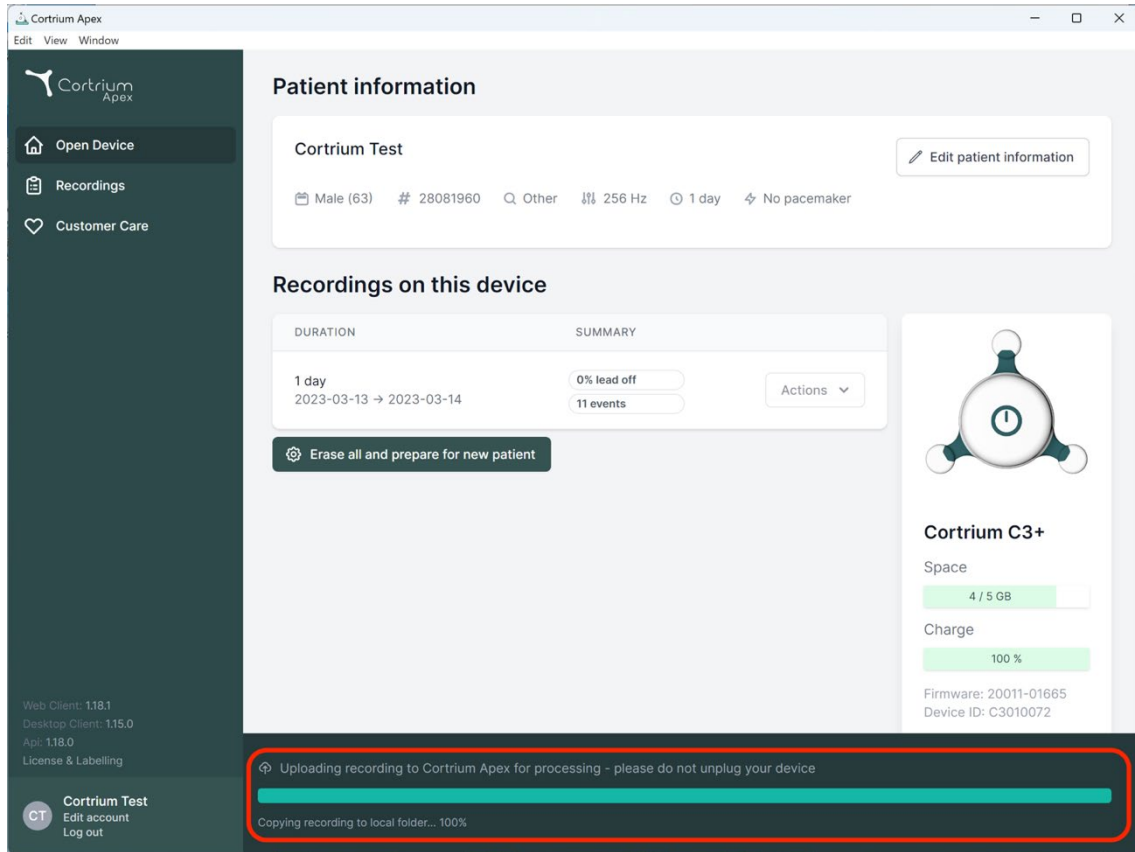
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and test duration (1 day). Below this, the 'Recordings on this device' section shows a table with columns for 'DURATION' and 'SUMMARY'. The table lists a recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. To the right, a device status card for 'Cortrium C3+' shows 'Space' at 4/5 GB and 'Charge' at 100%. At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' with a progress bar at 100% and the text 'Copying recording to local folder... 100%'.

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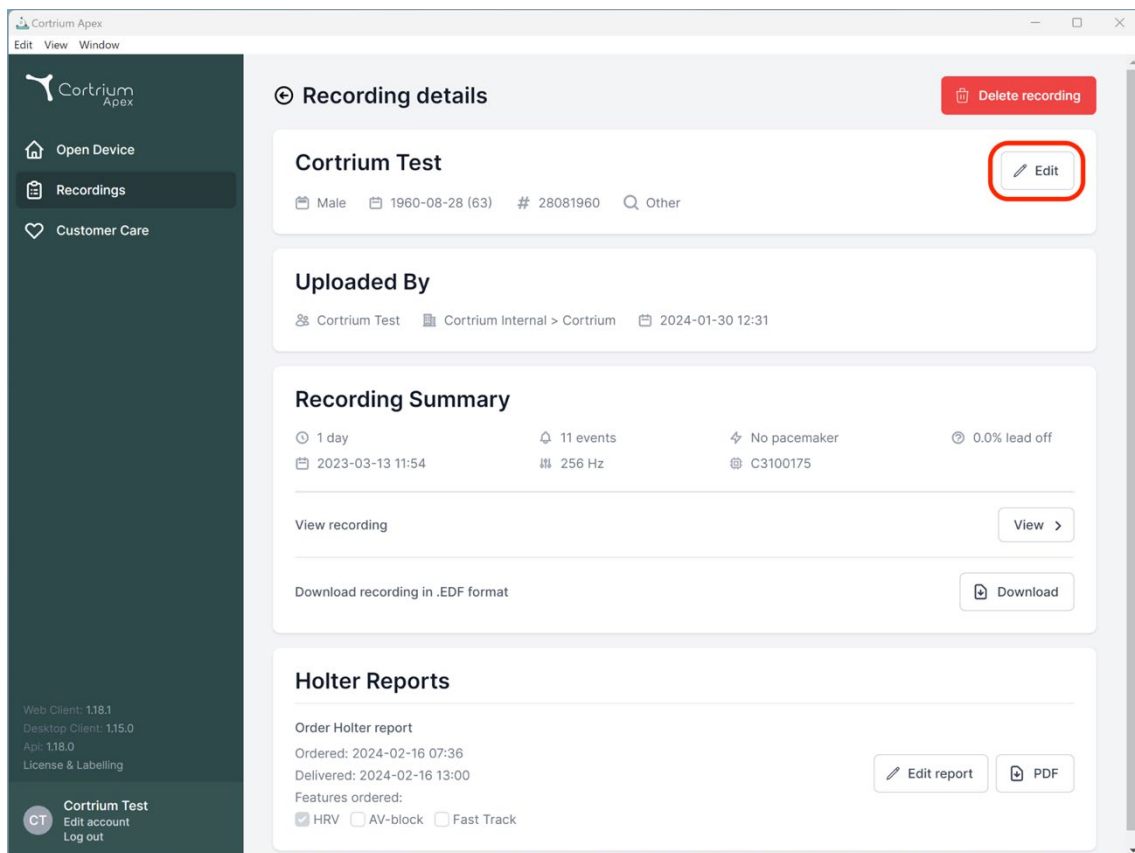
Cortrium Test
Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off

2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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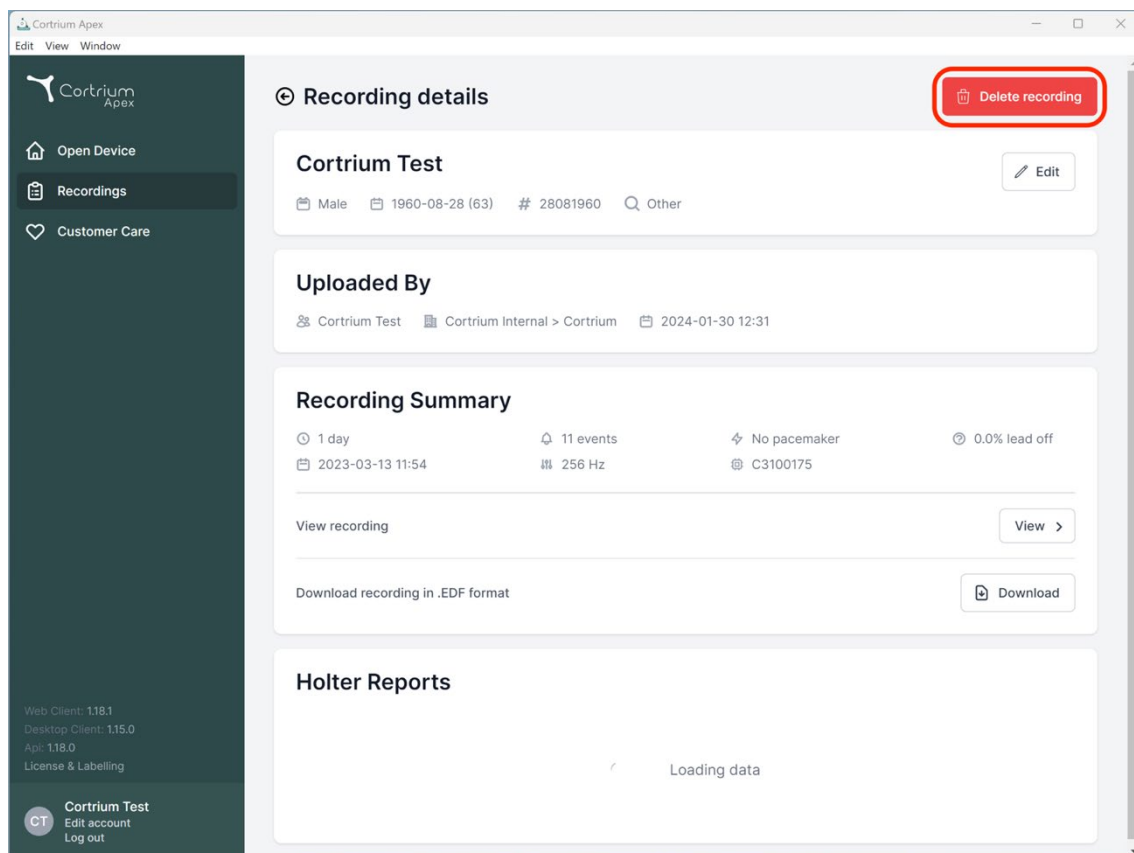
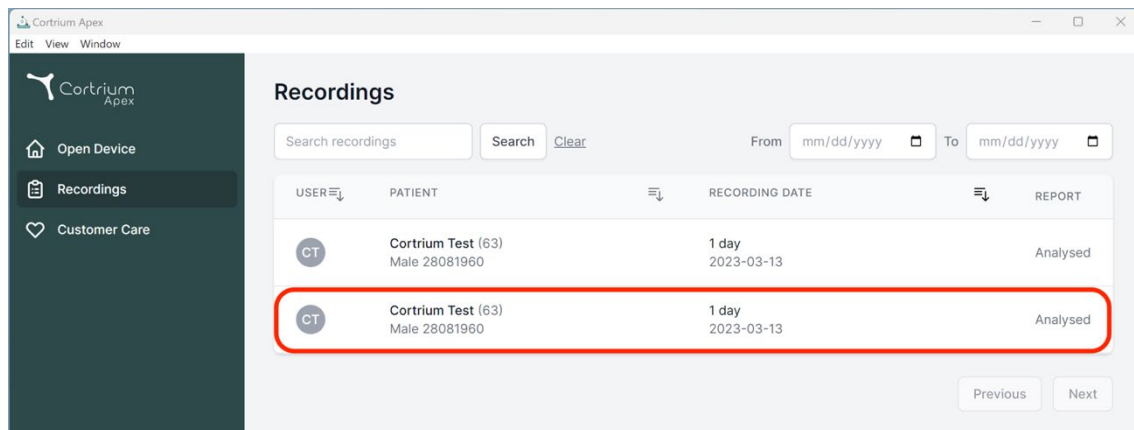
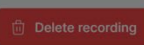
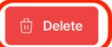
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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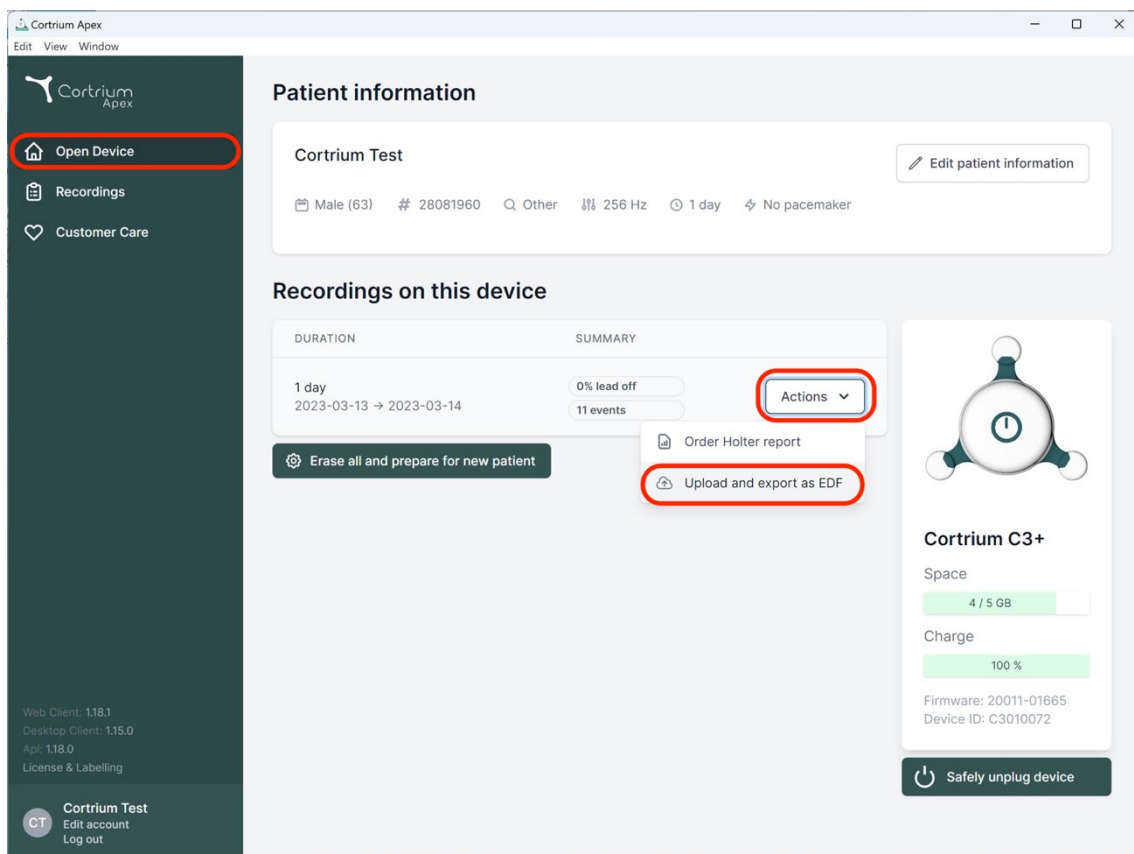
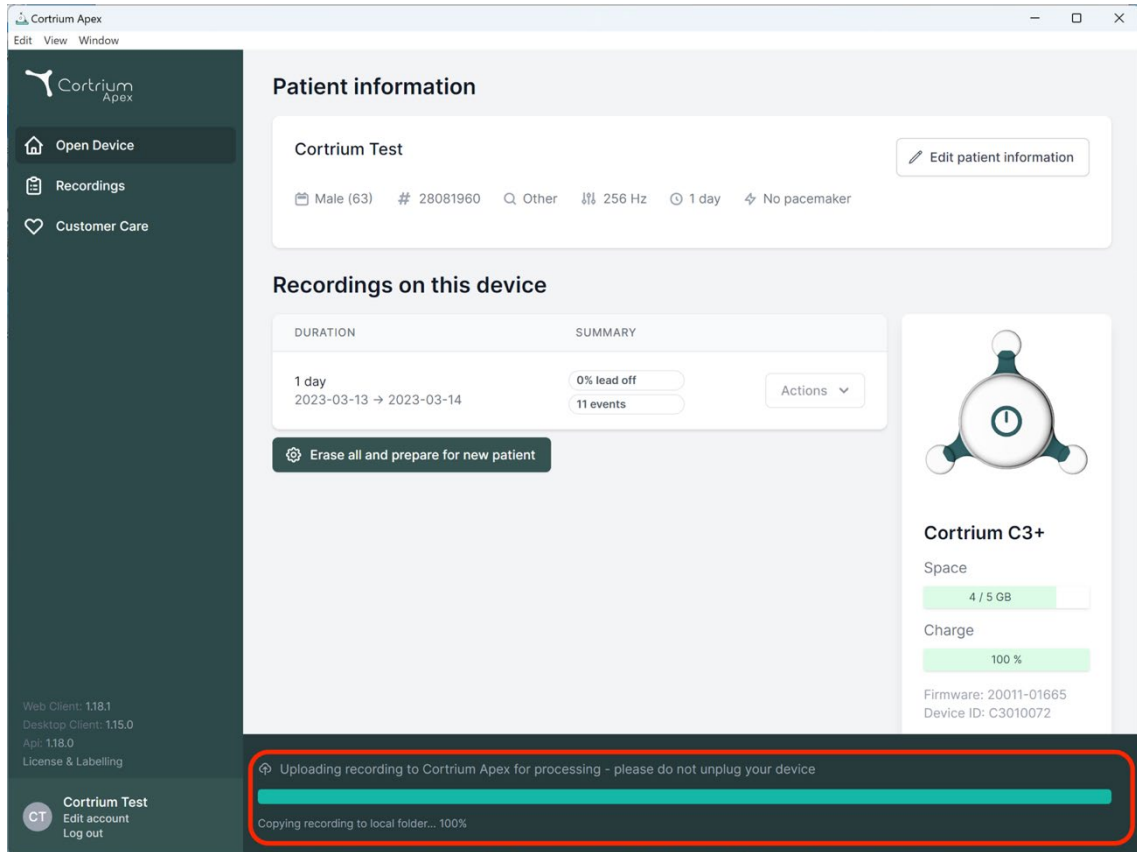
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

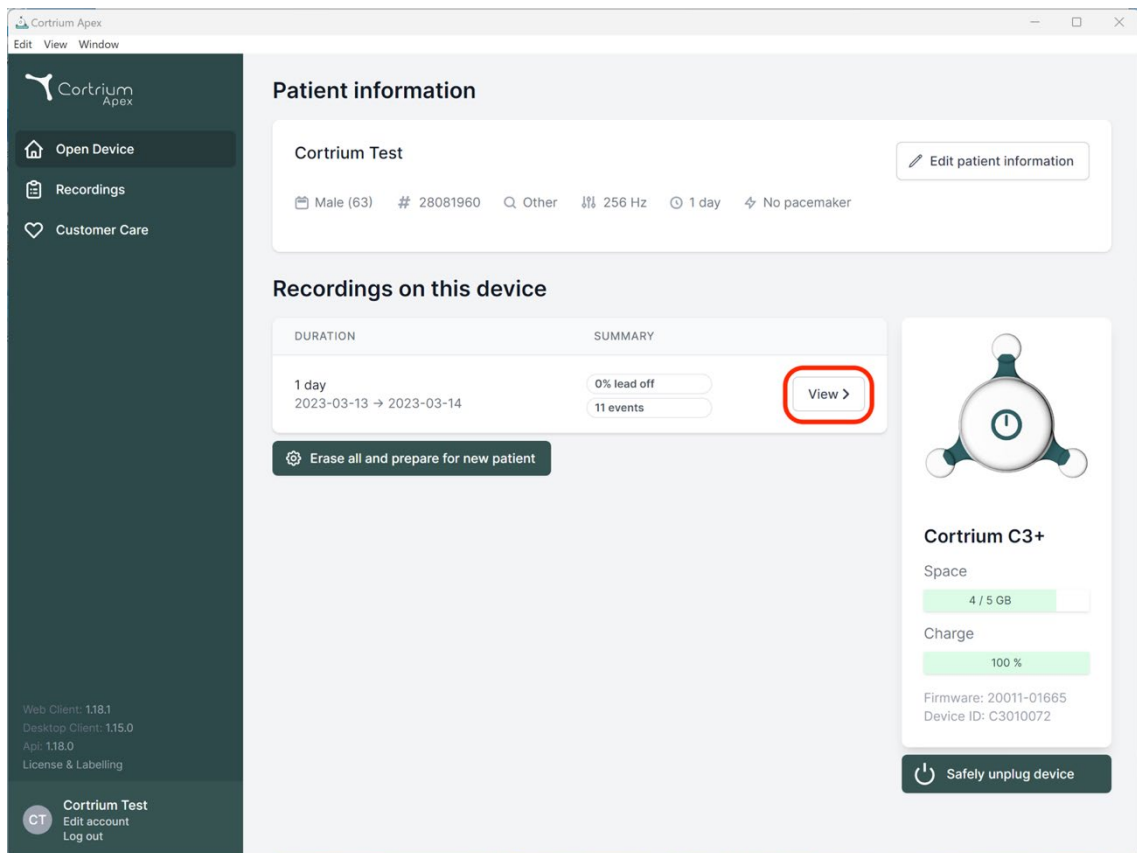
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
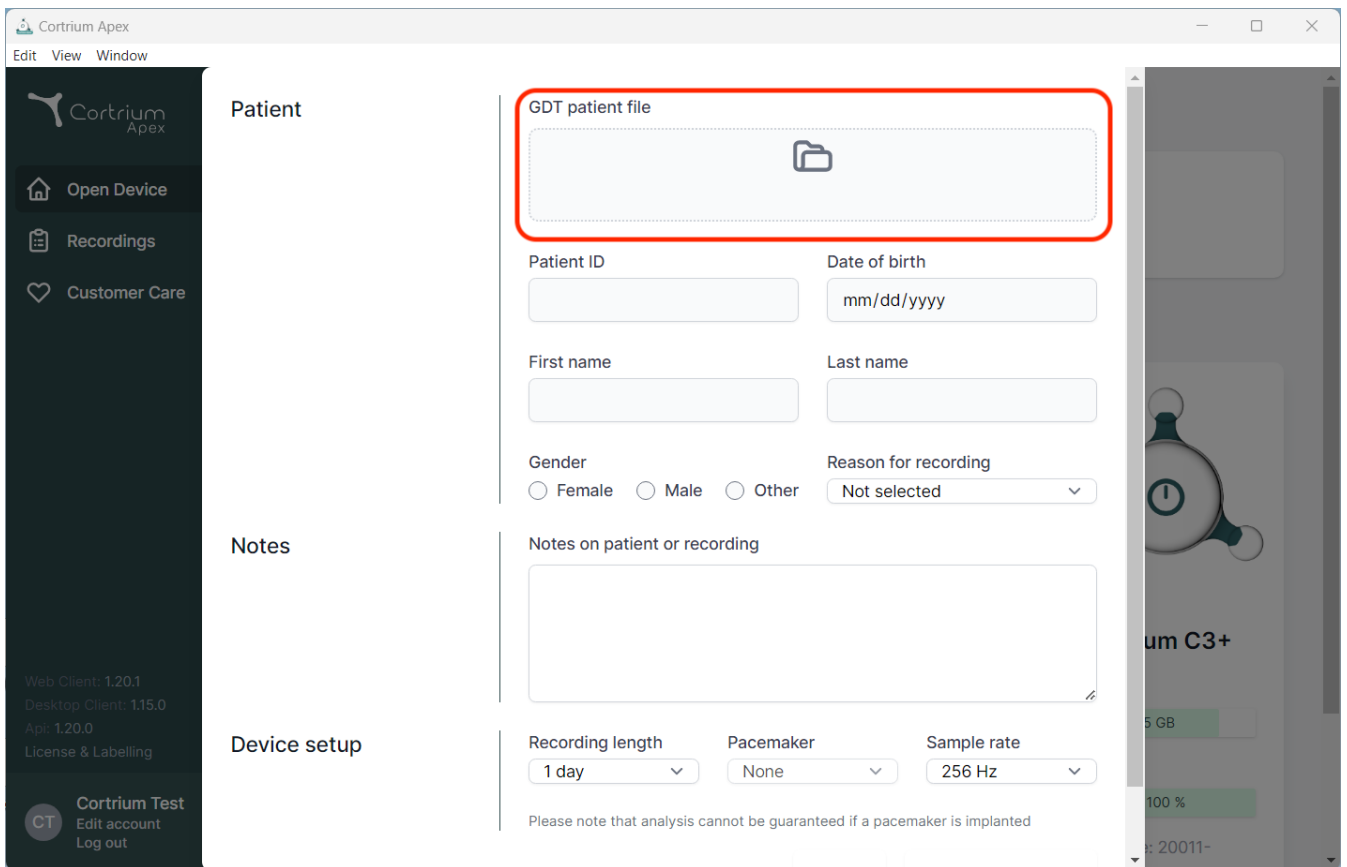
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: Open Device, Recordings, and Customer Care. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, App: 1.20.0) and a user profile section for Cortrium Test. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a red-bordered box labeled 'GDT patient file' with a folder icon, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdowns for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz), along with a disclaimer: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. On the right, a partial view of a device screen shows 'um C3+' and a battery level of 100%.

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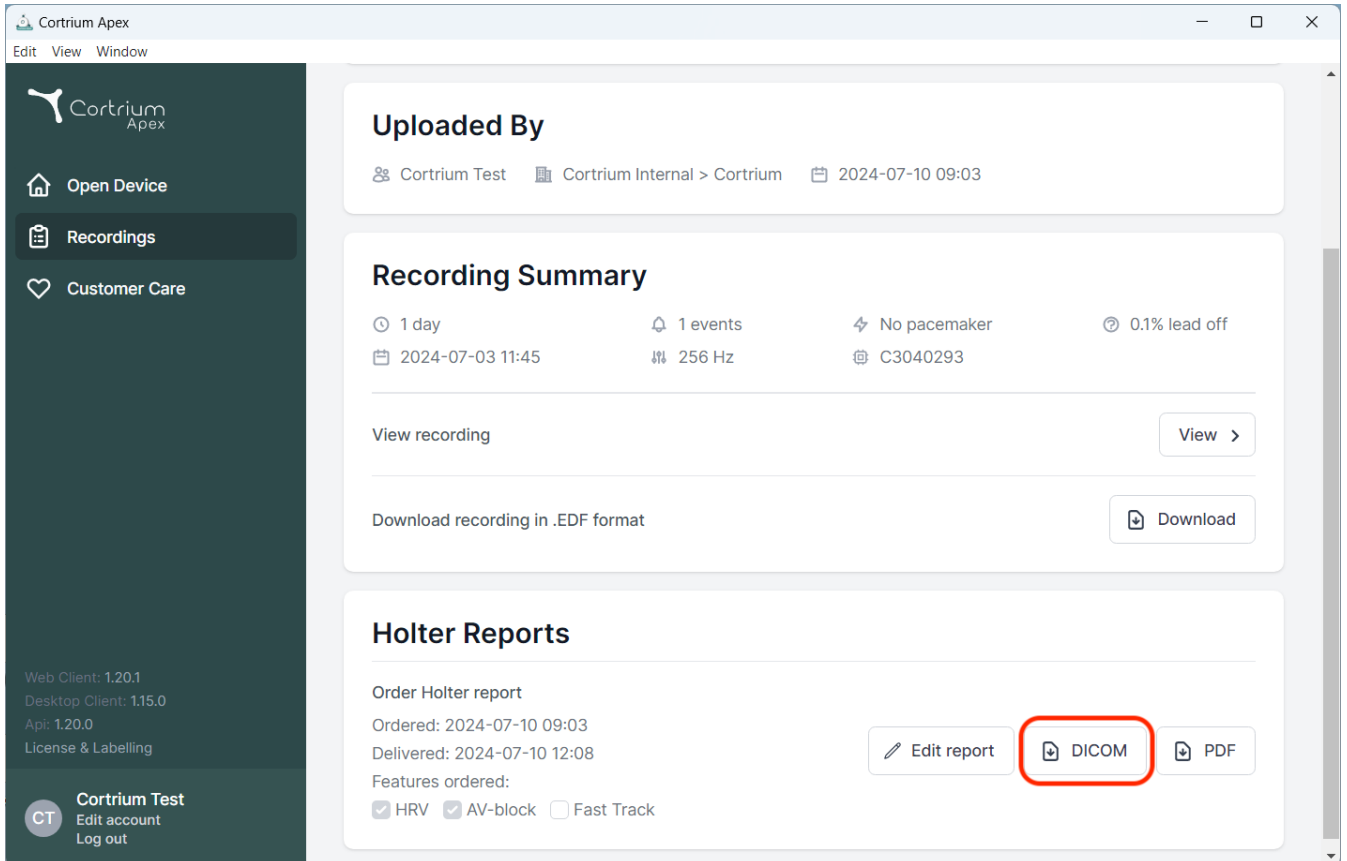
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', 'C3040293', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. The 'DICOM' button is highlighted with a red box.

The bottom left corner shows version information: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. The user 'Cortrium Test' is logged in.

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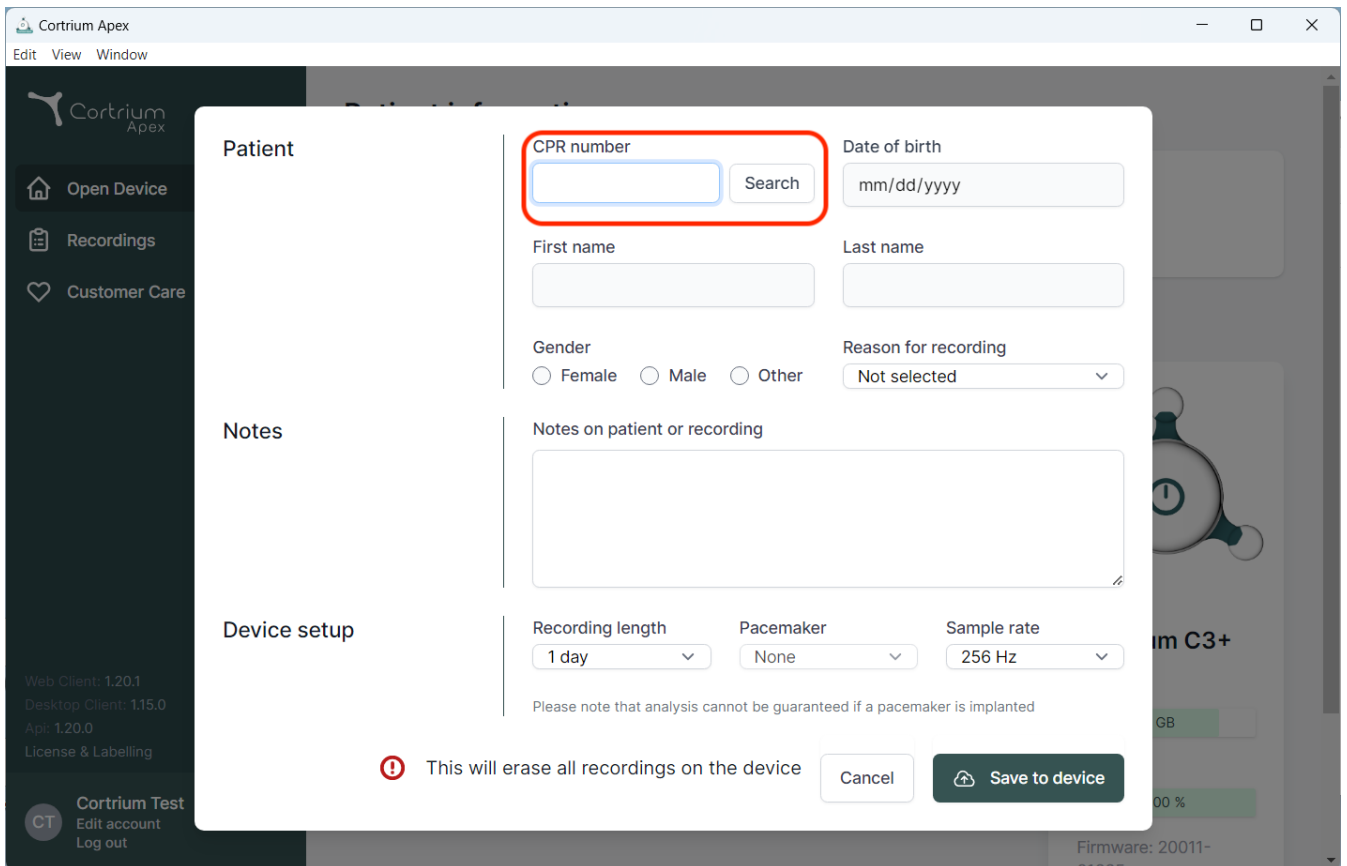The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient information. The "CPR number" field is highlighted with a red box, and a "Search" button is next to it. The form includes fields for "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), and "Notes on patient or recording". The "Device setup" section includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". Buttons for "Cancel" and "Save to device" are also present.

Patient

CPR number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length Pacemaker Sample rate

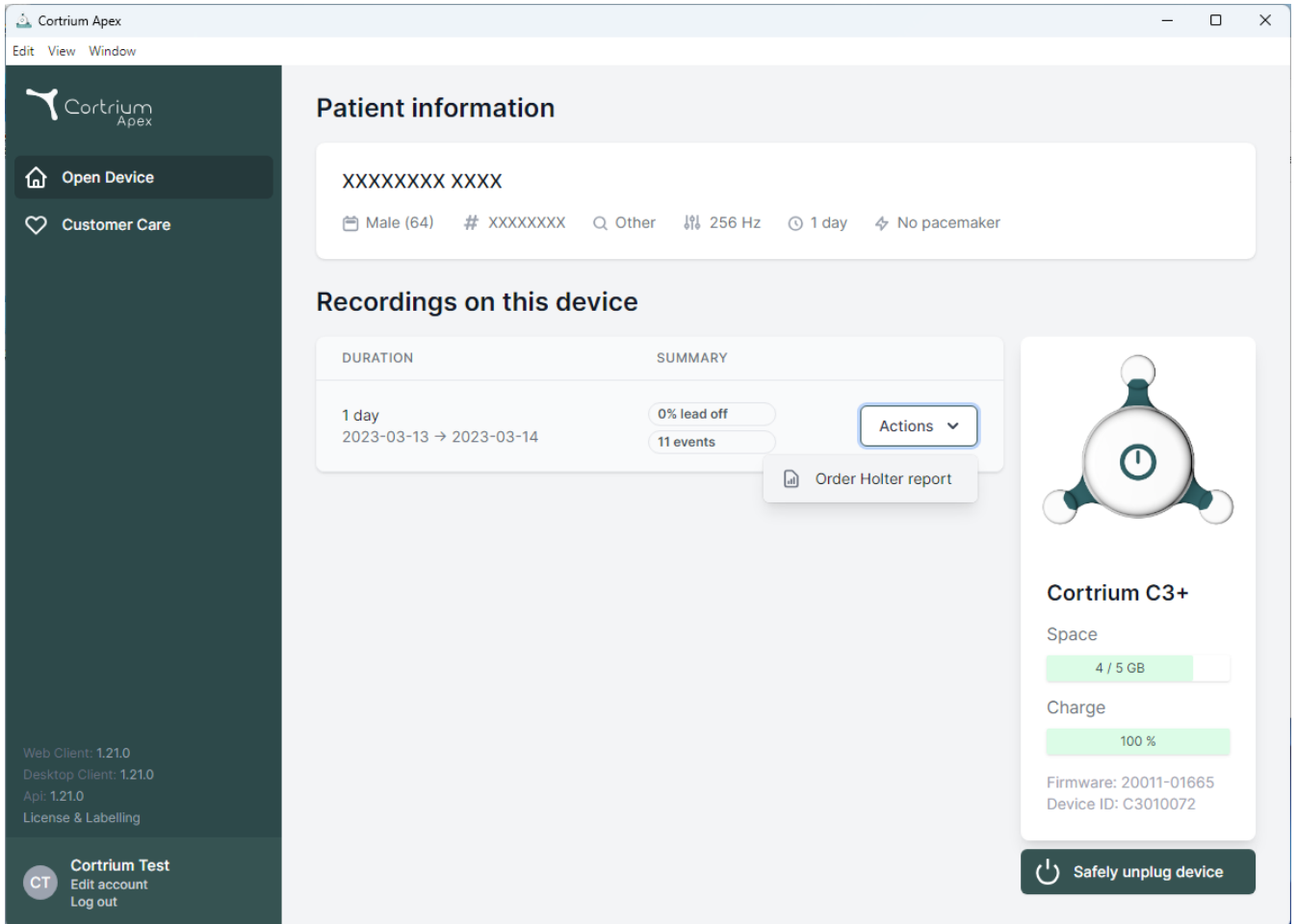
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), 'Customer Care' (with a heart icon), and version information: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar is a 'Cortrium Test' section with a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section contains a table with two columns: 'DURATION' and 'SUMMARY'. The table has one entry for '1 day' (2023-03-13 → 2023-03-14) with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space' (4 / 5 GB), 'Charge' (100 %), 'Firmware: 20011-01665', and 'Device ID: C3010072'. At the bottom of this card is a 'Safely unplug device' button.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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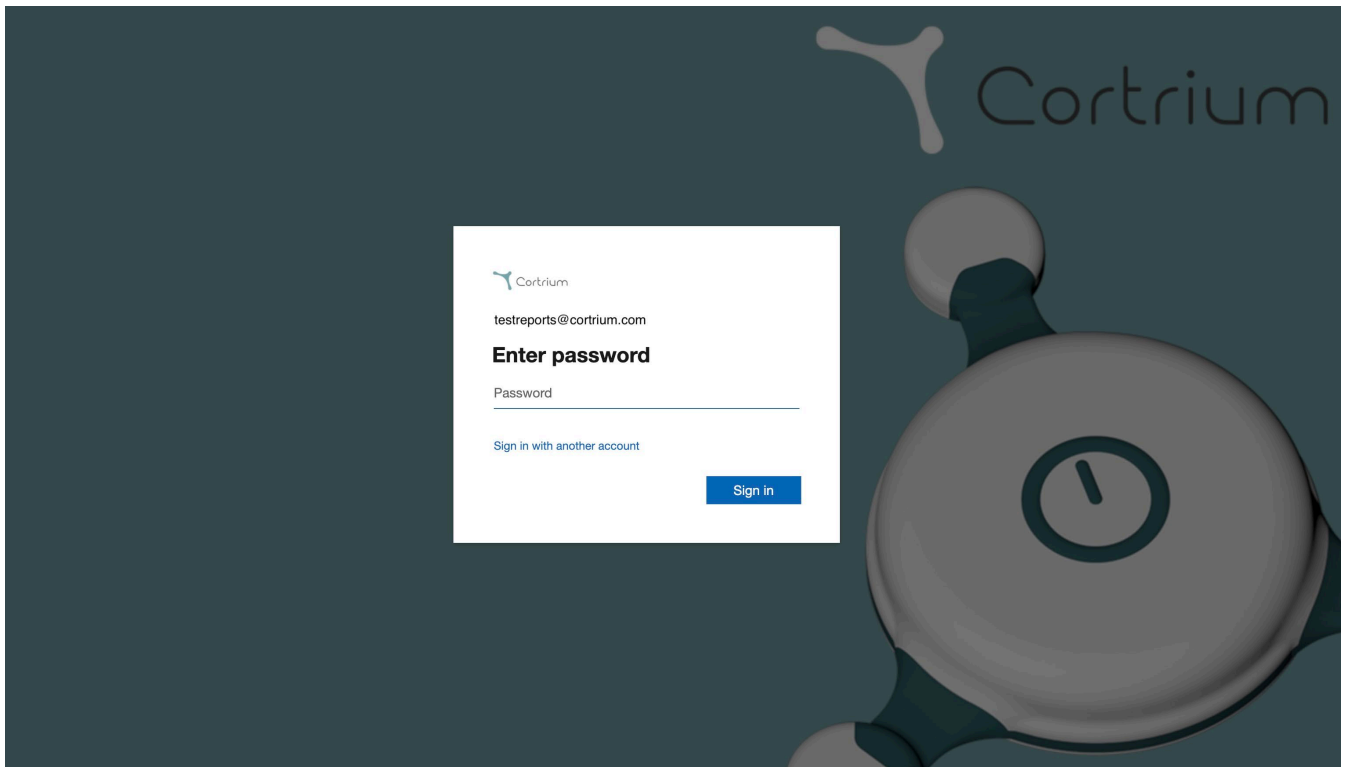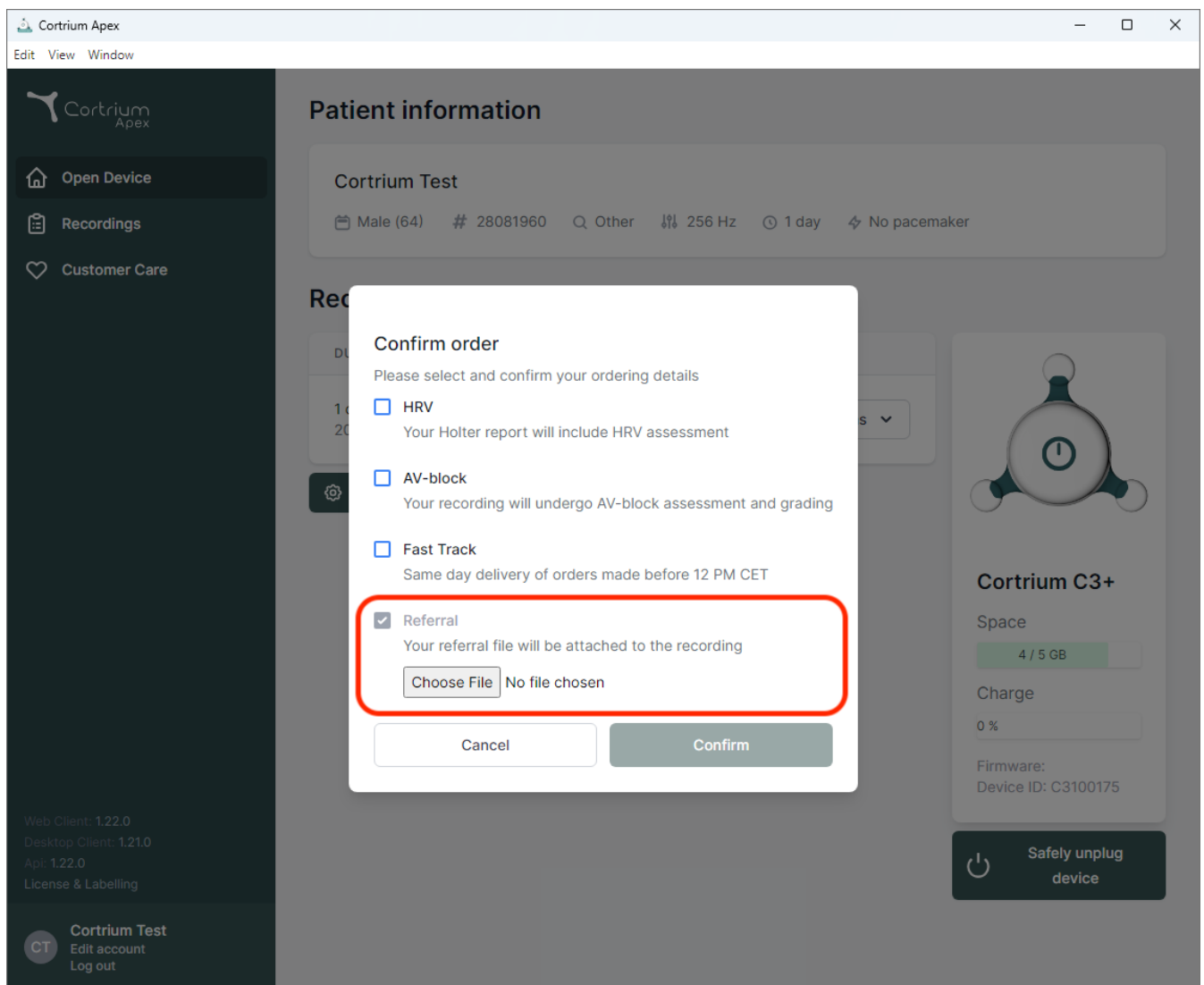
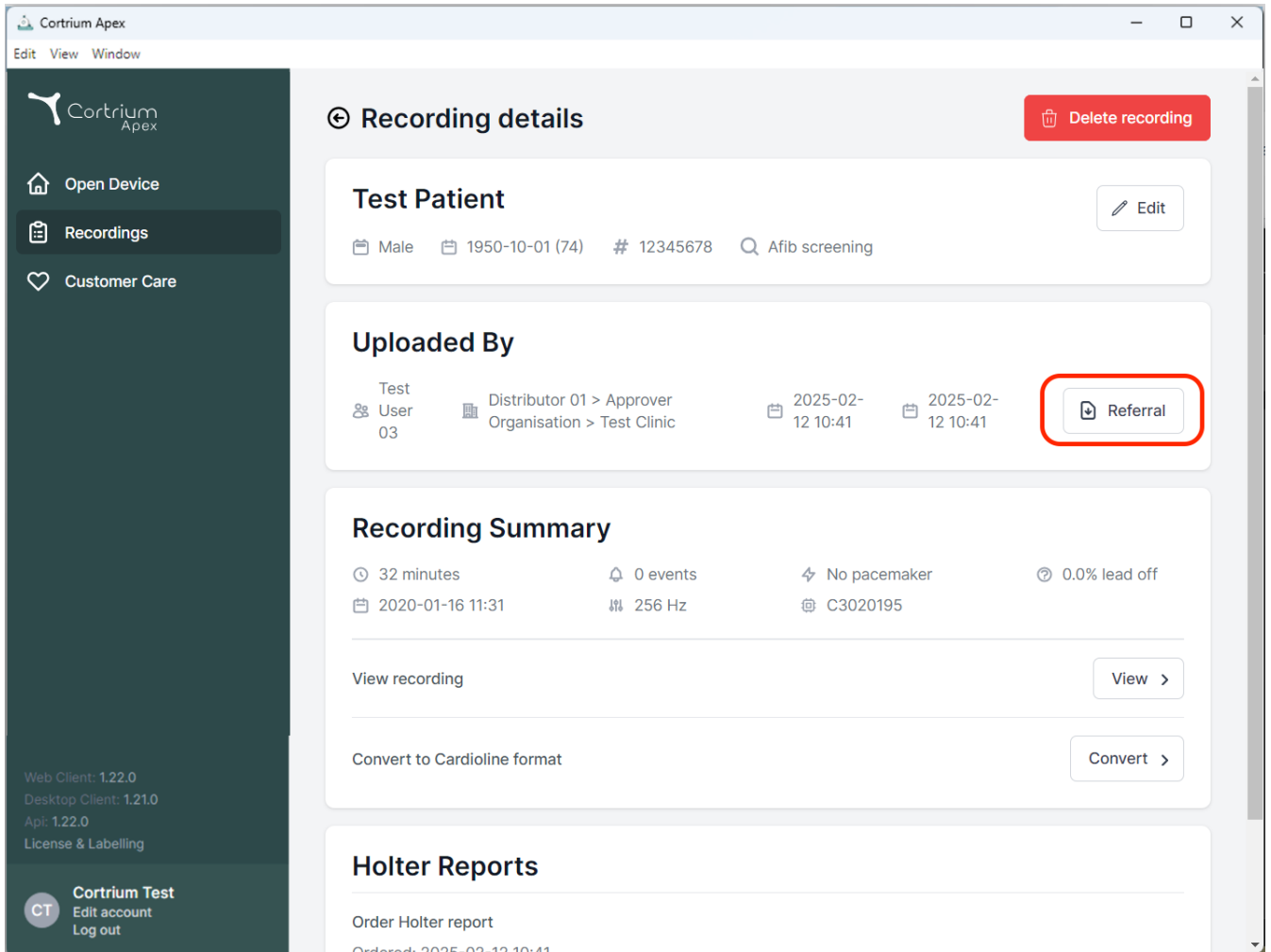
The screenshot displays the 'Recording details' page in the Cortrium Apex web application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right corner.

Test Patient section includes patient information: Male, 1950-10-01 (74), ID 12345678, and Afib screening. An 'Edit' button is available.

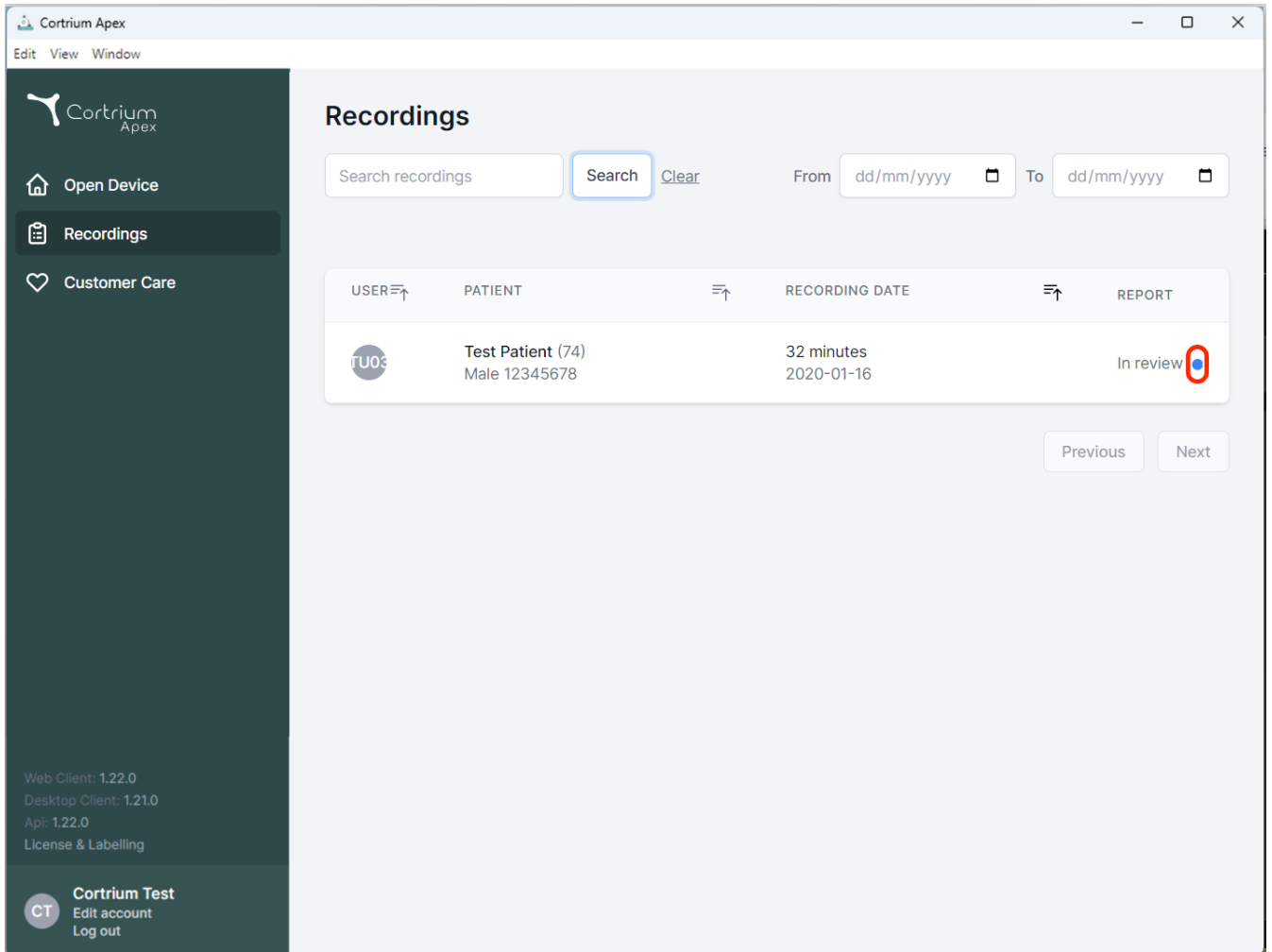
Uploaded By section shows the user 'User 03' and the distributor 'Distributor 01 > Approver Organisation > Test Clinic'. It also displays the upload date and time '2025-02-12 10:41'. A 'Referral' button, highlighted with a red box, is located in the top right of this section.

Recording Summary section provides key metrics: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date '2020-01-16 11:31', frequency '256 Hz', and device ID 'C3020195'. Below this, there are buttons for 'View recording' and 'Convert to Cardioline format'.

Holter Reports section includes an 'Order Holter report' button and a note 'Ordered: 2025-02-12 10:41'.

The bottom of the sidebar contains version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

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
The screenshot shows the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area displays a table of recordings. The table has columns for 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. A single recording is listed for 'Test Patient (74)' with a recording date of '2020-01-16' and a duration of '32 minutes'. The 'REPORT' column shows 'In review' with a blue dot icon circled in red, indicating that the report is not yet finalized. Below the table are 'Previous' and 'Next' navigation buttons. The bottom of the sidebar contains version information and a user profile section for 'Cortrium Test'.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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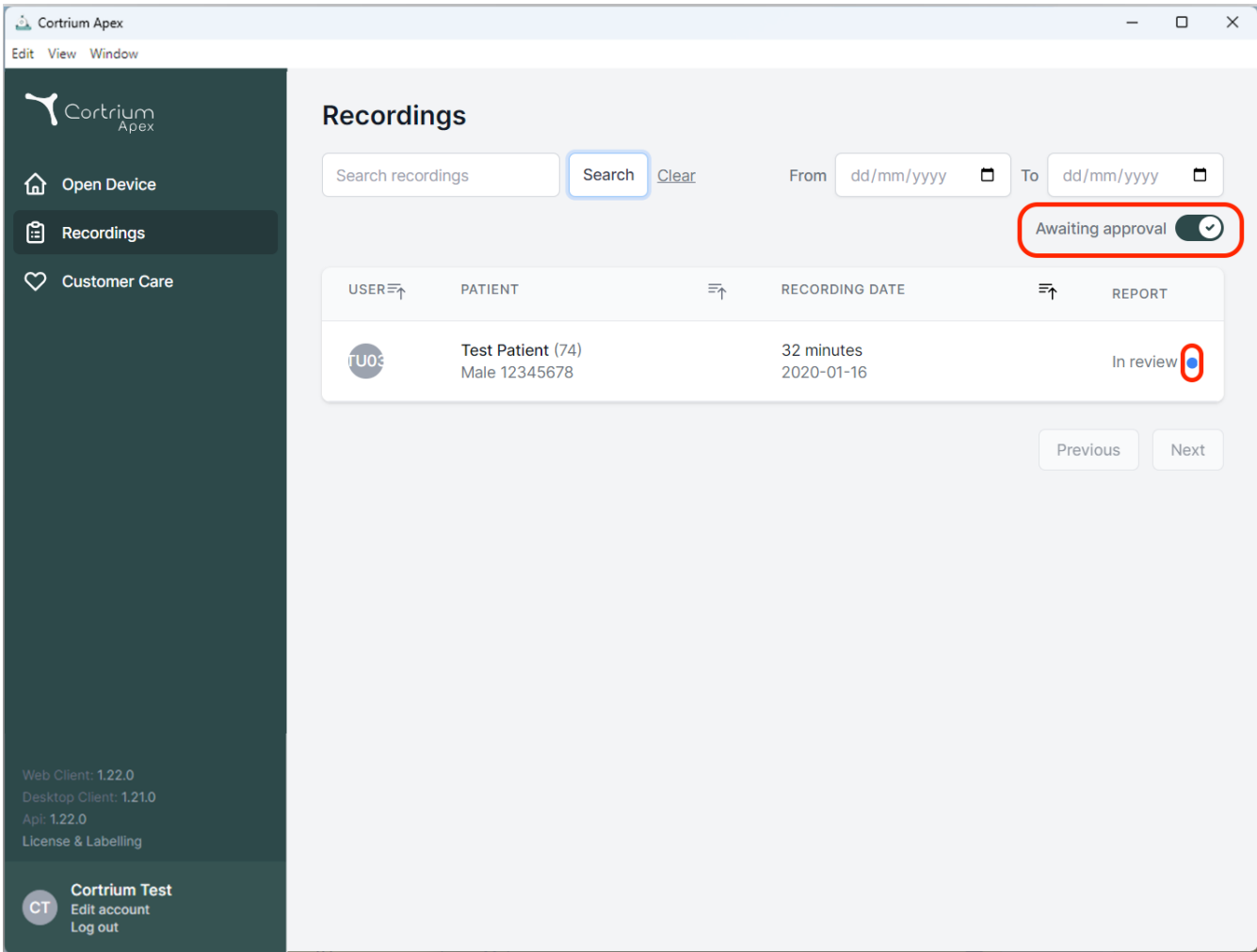
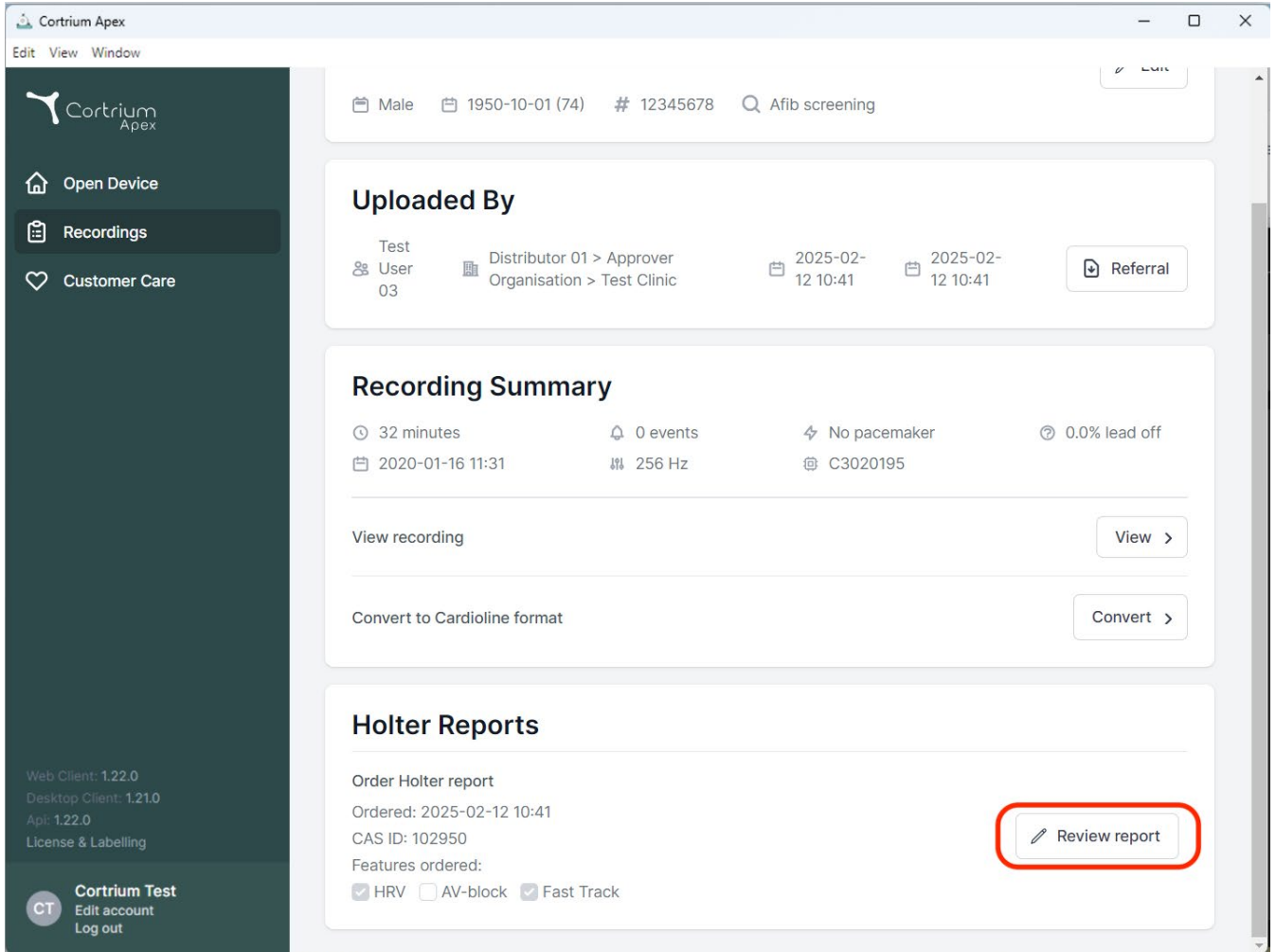
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A red box highlights the 'Review report' button.

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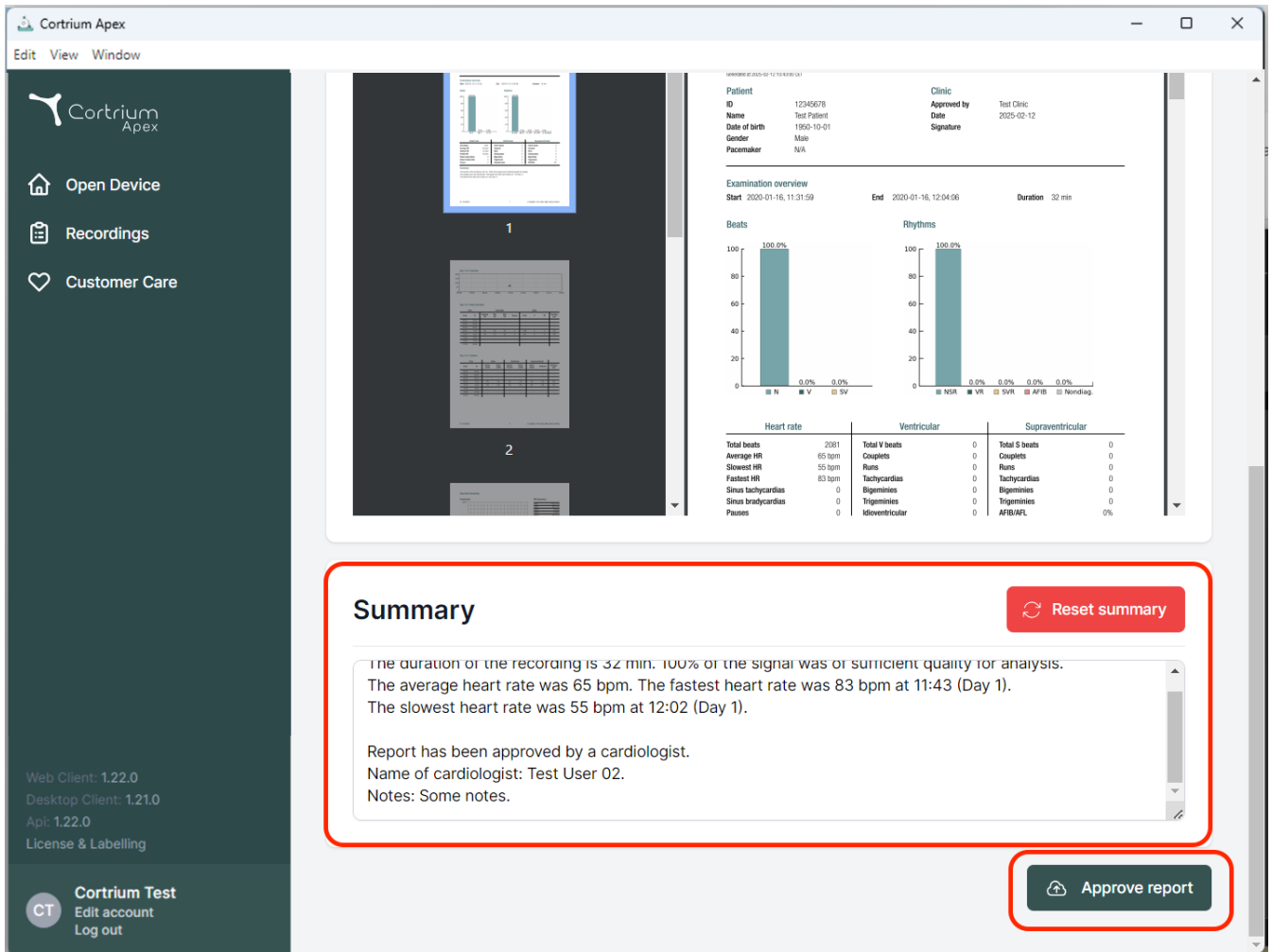Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

Beats
100.0%
0.0% 0.0% 0.0%

Rhythms
100.0%
0.0% 0.0% 0.0% 0.0% 0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AFIB/AFI	0%

01	First version in SimplerQMS based on first external release of Apex
02	Updated in relation to version 12 of Apex for the recording viewer
03	Added warning section as risk mitigation related to cyber security. Added Labelling information under point 14. Added CE mark at bottom of Document.
04	Added security information section 14 (labelling is now 15)
05	Updates for Apex 1.16 Added two points to Section 6.2 and exchanged the screenshot of the recording viewer to describe the amplitude zoom and endless scrolling functionality Updated screenshots in point 4 and 7 related to setting up device and editing patient info Added UDI and manufacturing site to last page for better linkage to product Added missing sentence in section 16 regarding port 443 and updated URLs
06	Updated label with “MD” and Address and updated Apex to 1.17
07	Updated with new screenshots to reflect latest changes, and new add-on features. This has affected the entire document but new text has been added in Section 5 In section 1, an extra section has been added describing discrepancies in versioning vs screenshots.
08	Added screen shot and grading information to AV Block add on in section 5
09	Added Appendix I and a reference to this under point 1. Release date added to last page.
10	Release date added to last page.
11	Added Uploader only role and AD integration to Appendix and change system requirements for macOS to Big Sur (as part of Apex 1.21)
12	Update to APEX to version 1.22 Appendix vii Added integration for 3party Approval flow to APEX for distributors and organisations Appendix viii Added Integration for requester to attach patient referral

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033