Cortrium Apex 1.22

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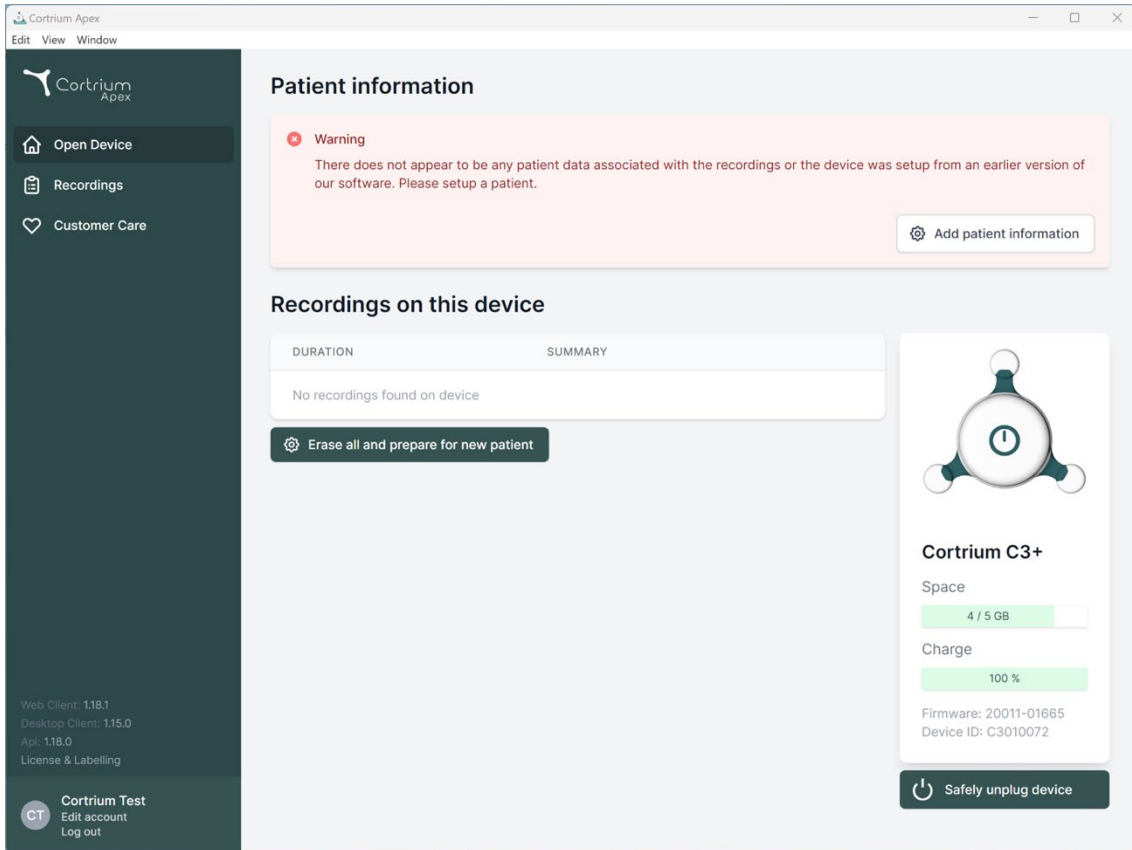
Patient information

Warning

There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.

[Add patient information](#)

Recordings on this device

DURATION	SUMMARY
No recordings found on device	

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Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning This will erase all recordings on the device

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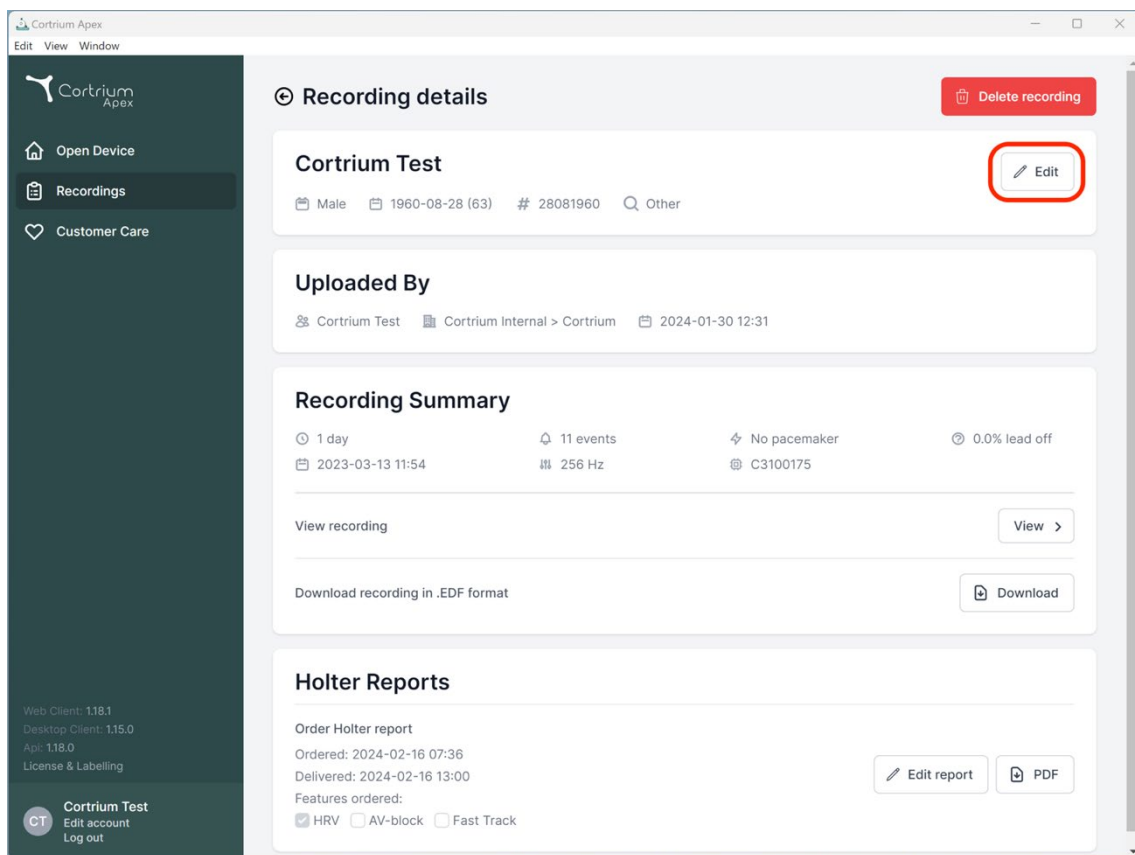
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
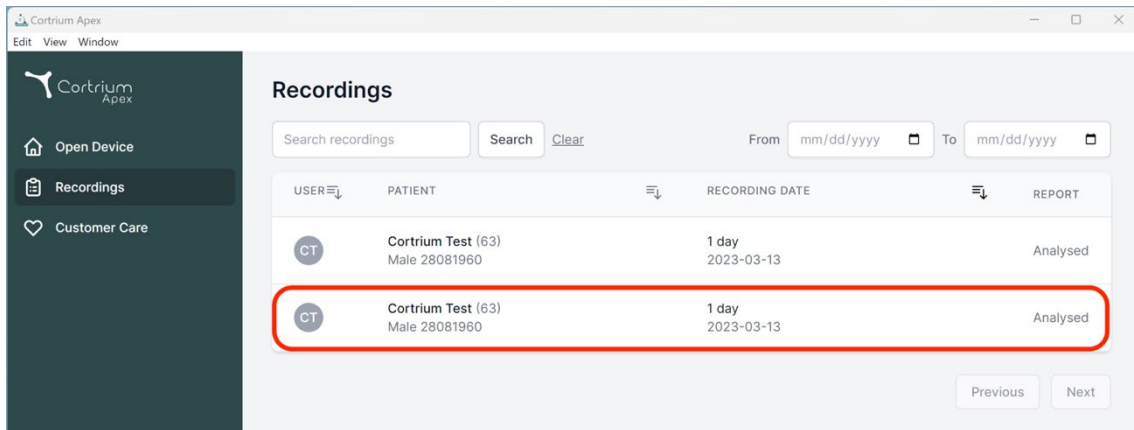
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. On the left is a sidebar with 'Open Device', 'Recordings', and 'Customer Care'. The main area has a search bar and a table of recordings. The table has columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red box.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

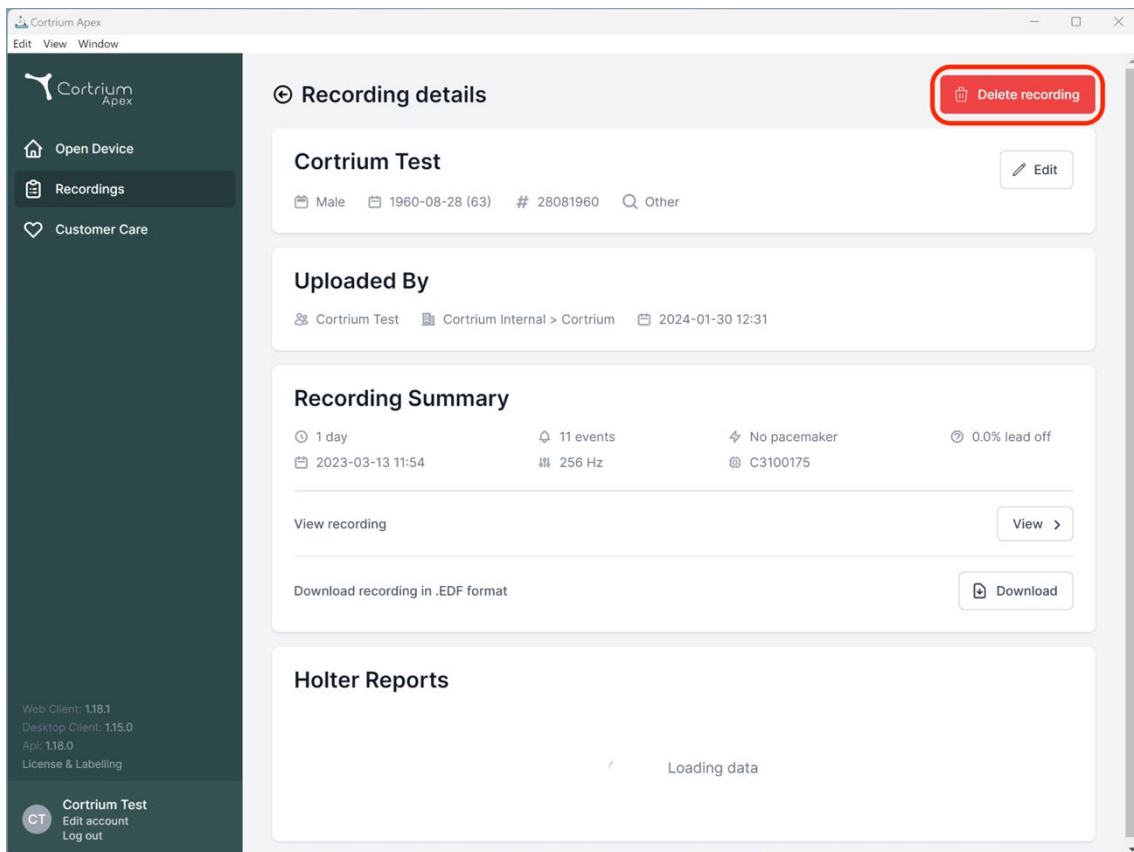The screenshot shows the 'Recording details' page for 'Cortrium Test'. The page has a sidebar with 'Open Device', 'Recordings', and 'Customer Care'. The main area shows the recording details, including a 'Delete recording' button highlighted with a red box. The recording details are: 1 day, 11 events, No pacemaker, 0.0% lead off, 2023-03-13 11:54, 256 Hz, C3100175.

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Loading data

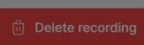
Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Recording details 

Cortrium Test 

 Male  1960-08-28 (63)  # 28081960  Other

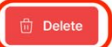
Uploaded By


 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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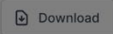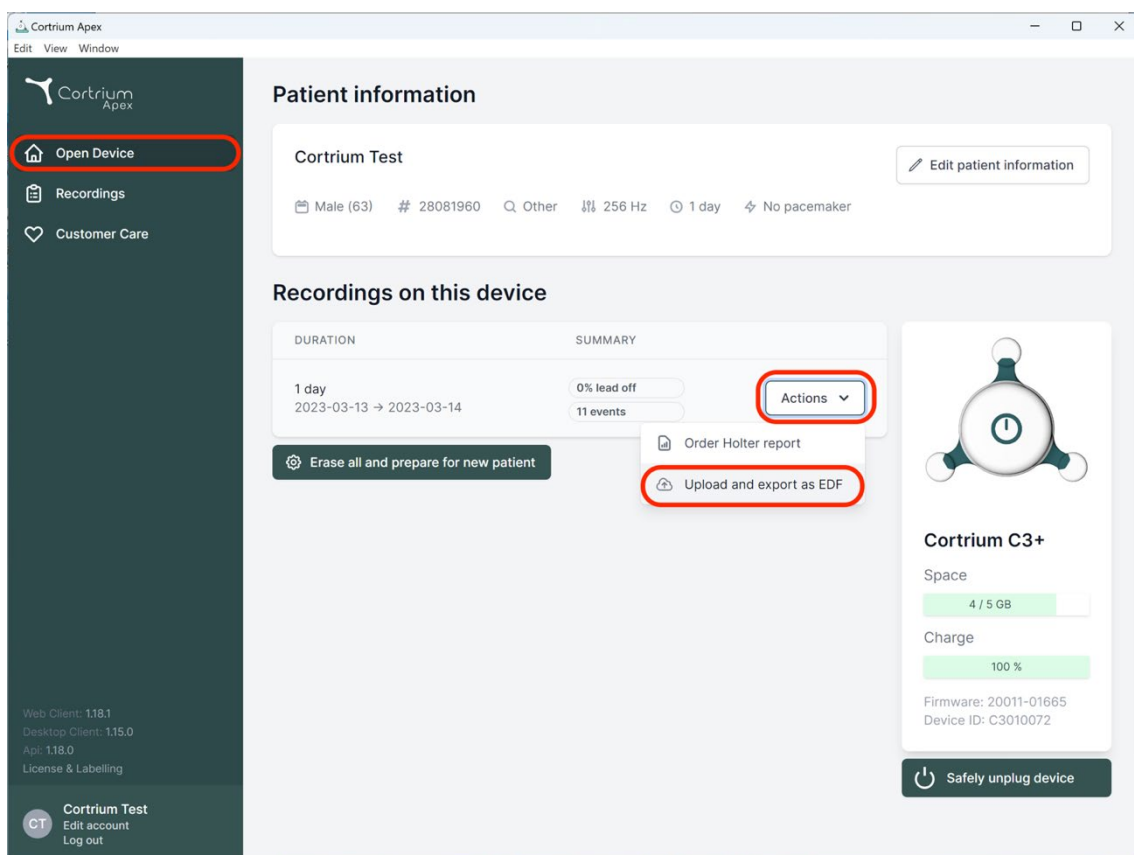
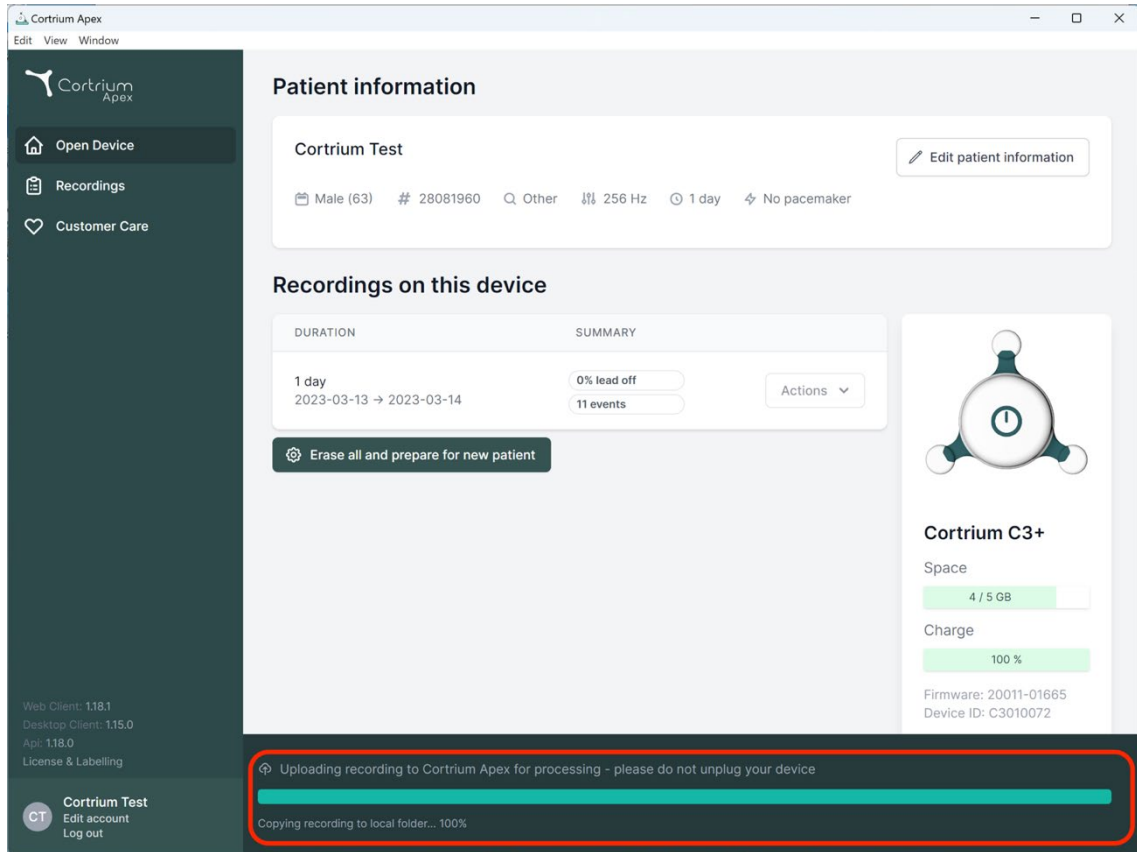
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

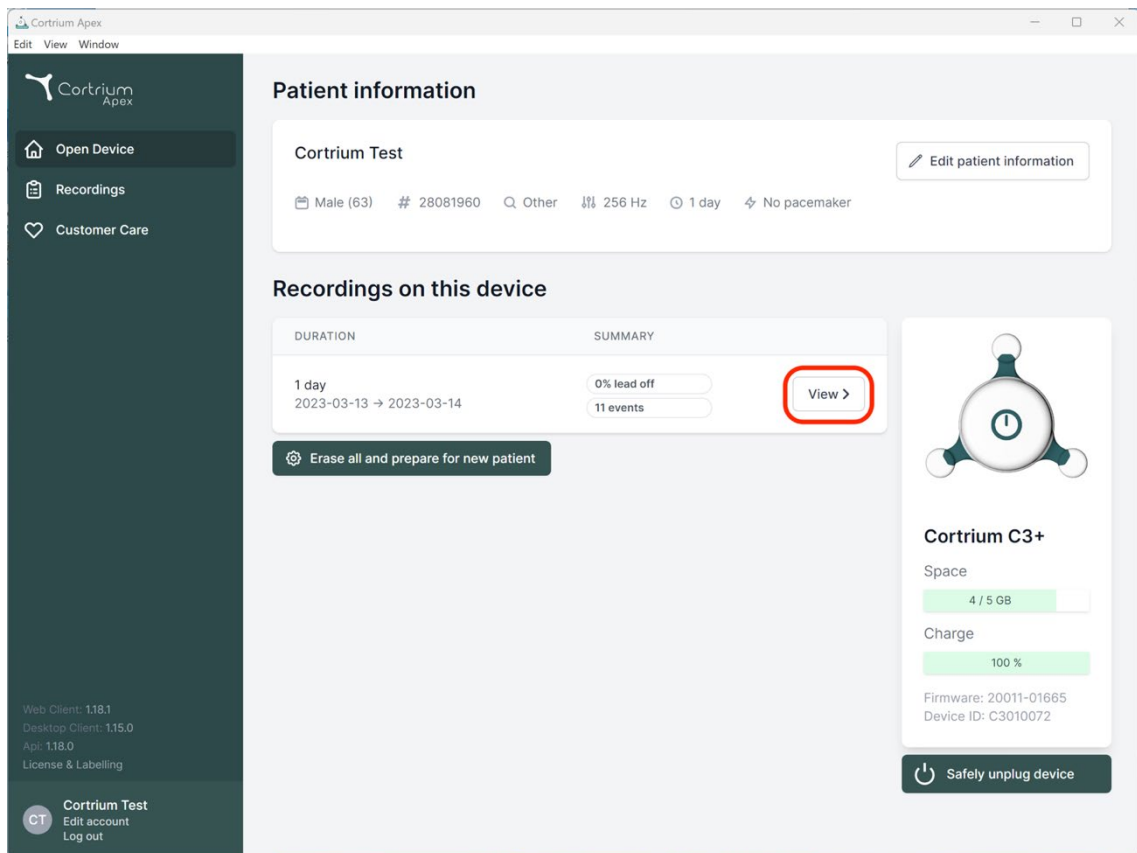
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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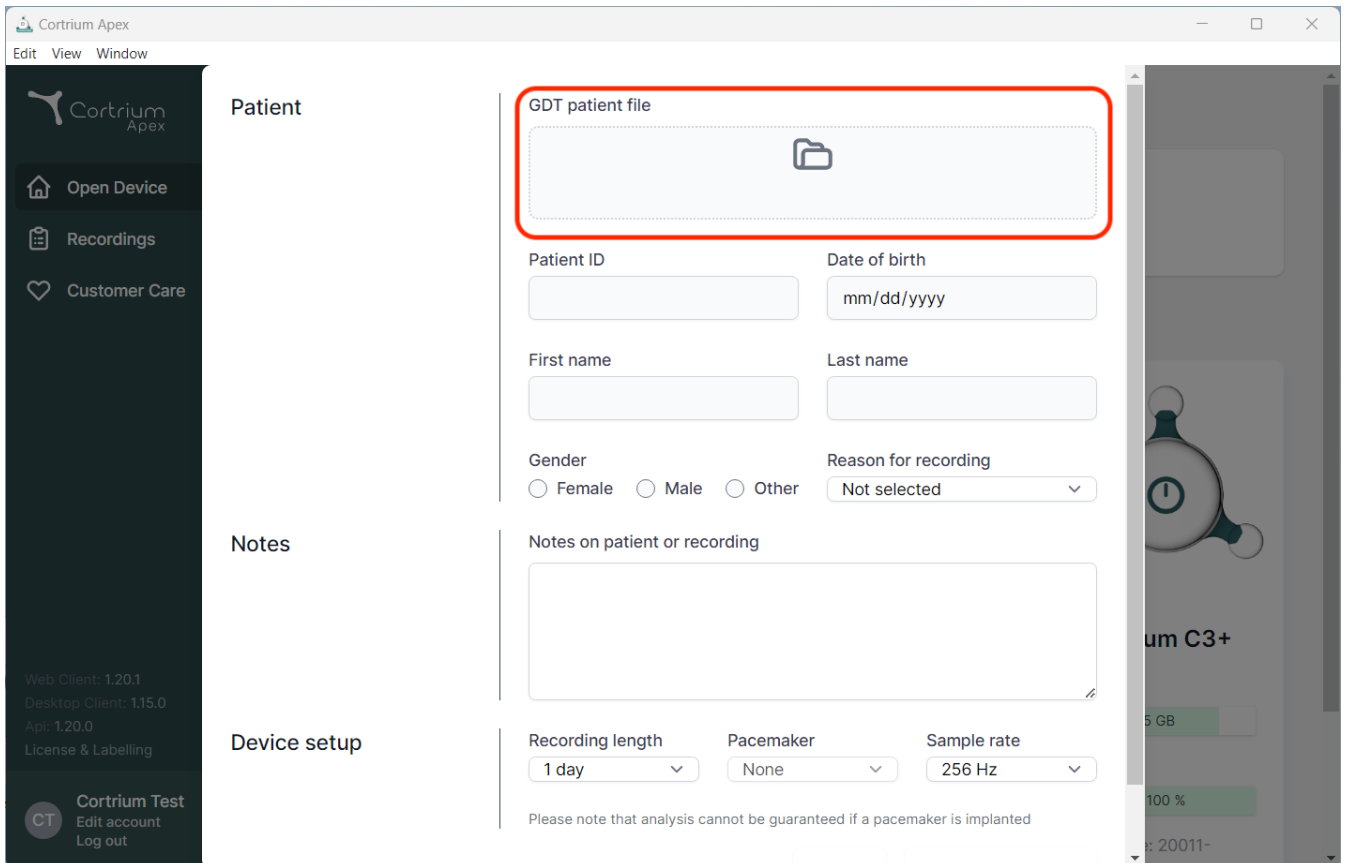
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted and contains a form for entering patient information. The form includes a red-bordered box for the GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. Below these are radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

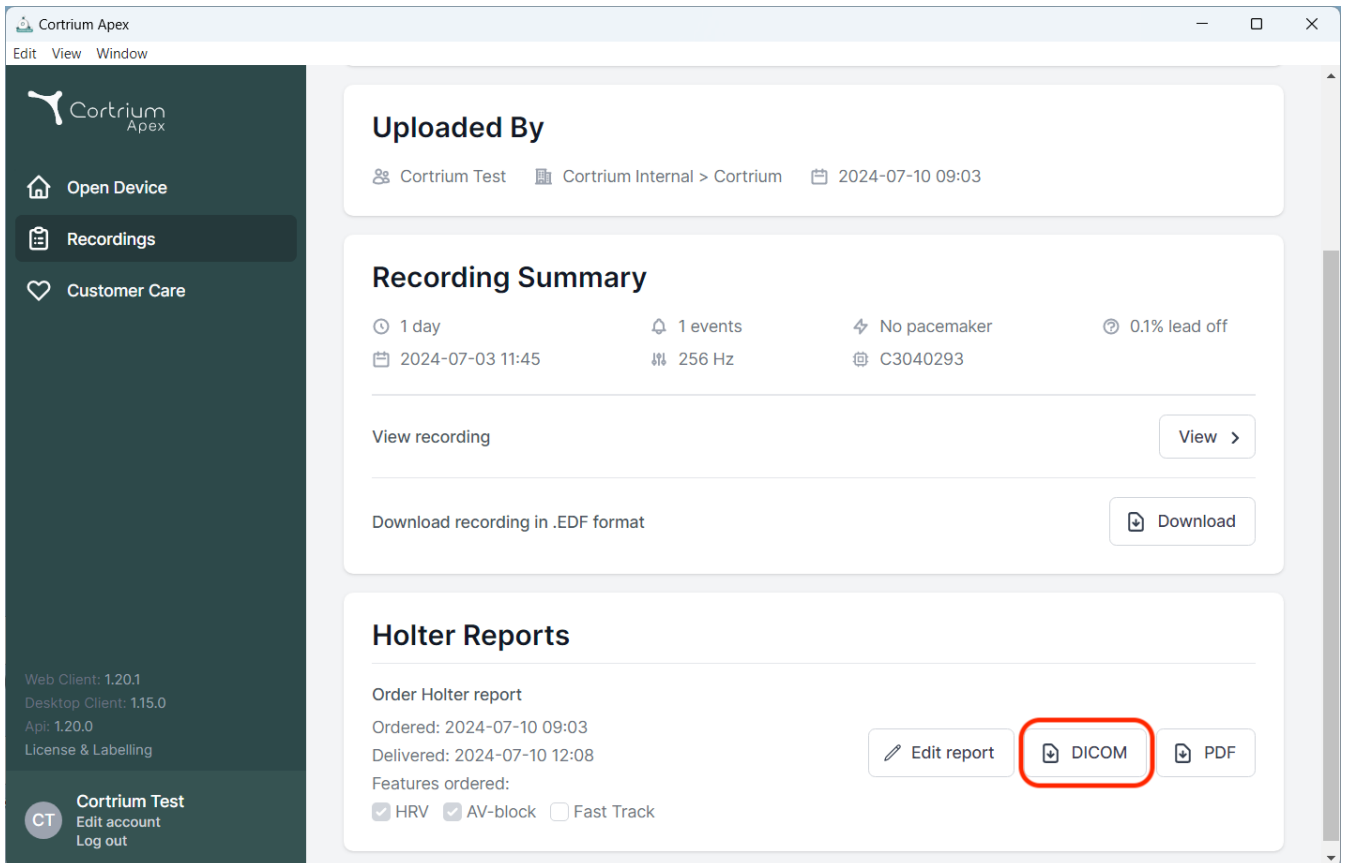
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom left of the sidebar shows version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with a 'Log out' link.

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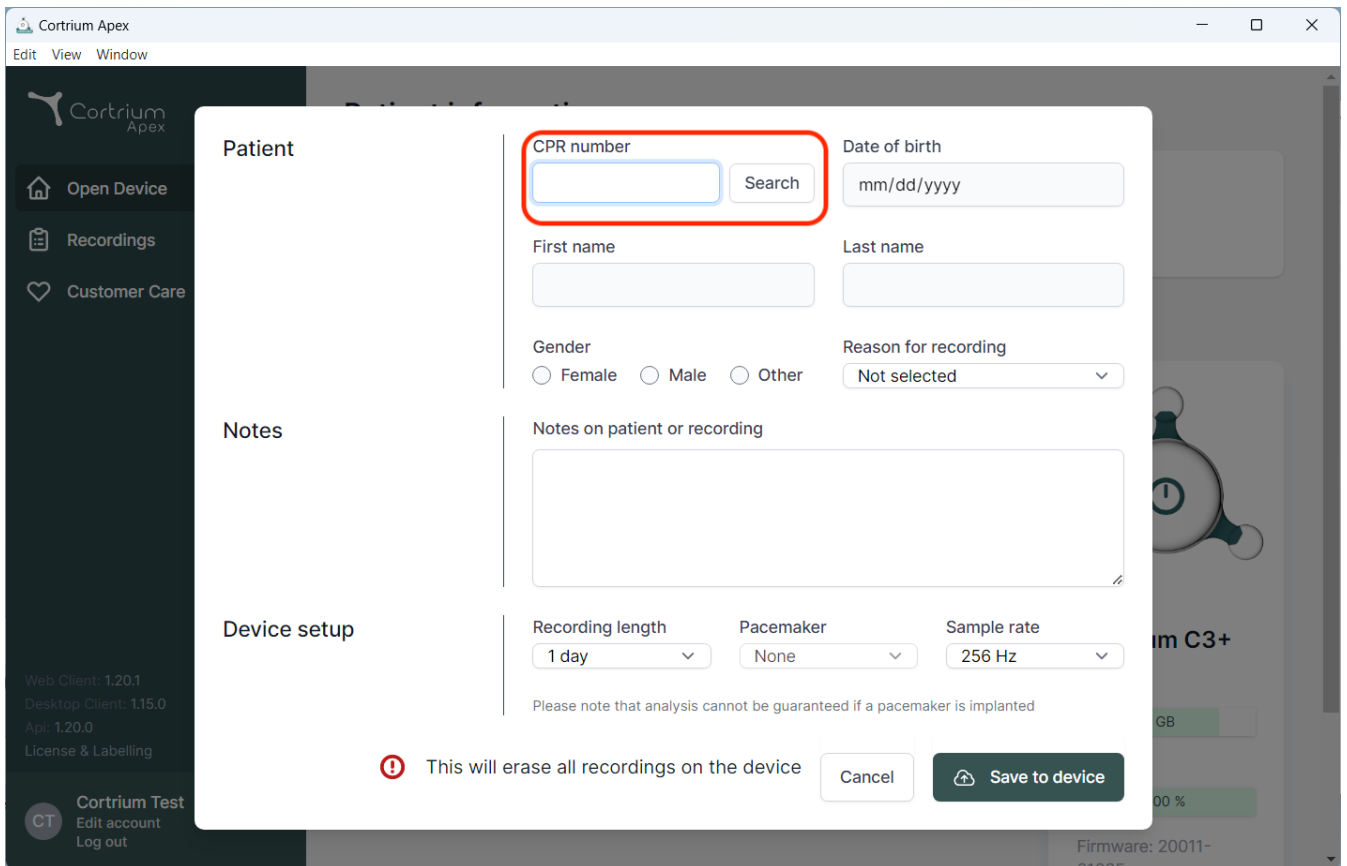The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for searching and entering patient information. The form includes fields for CPR number (highlighted with a red box), Date of birth, First name, Last name, Gender (Female, Male, Other), Reason for recording, and a large text area for Notes. Below the notes section is the "Device setup" section with dropdowns for Recording length, Pacemaker, and Sample rate. At the bottom of the modal, there is a warning message: "This will erase all recordings on the device" with a red exclamation mark icon, and two buttons: "Cancel" and "Save to device". The background shows the main application menu with options like "Open Device", "Recordings", and "Customer Care".

Patient

CPR number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

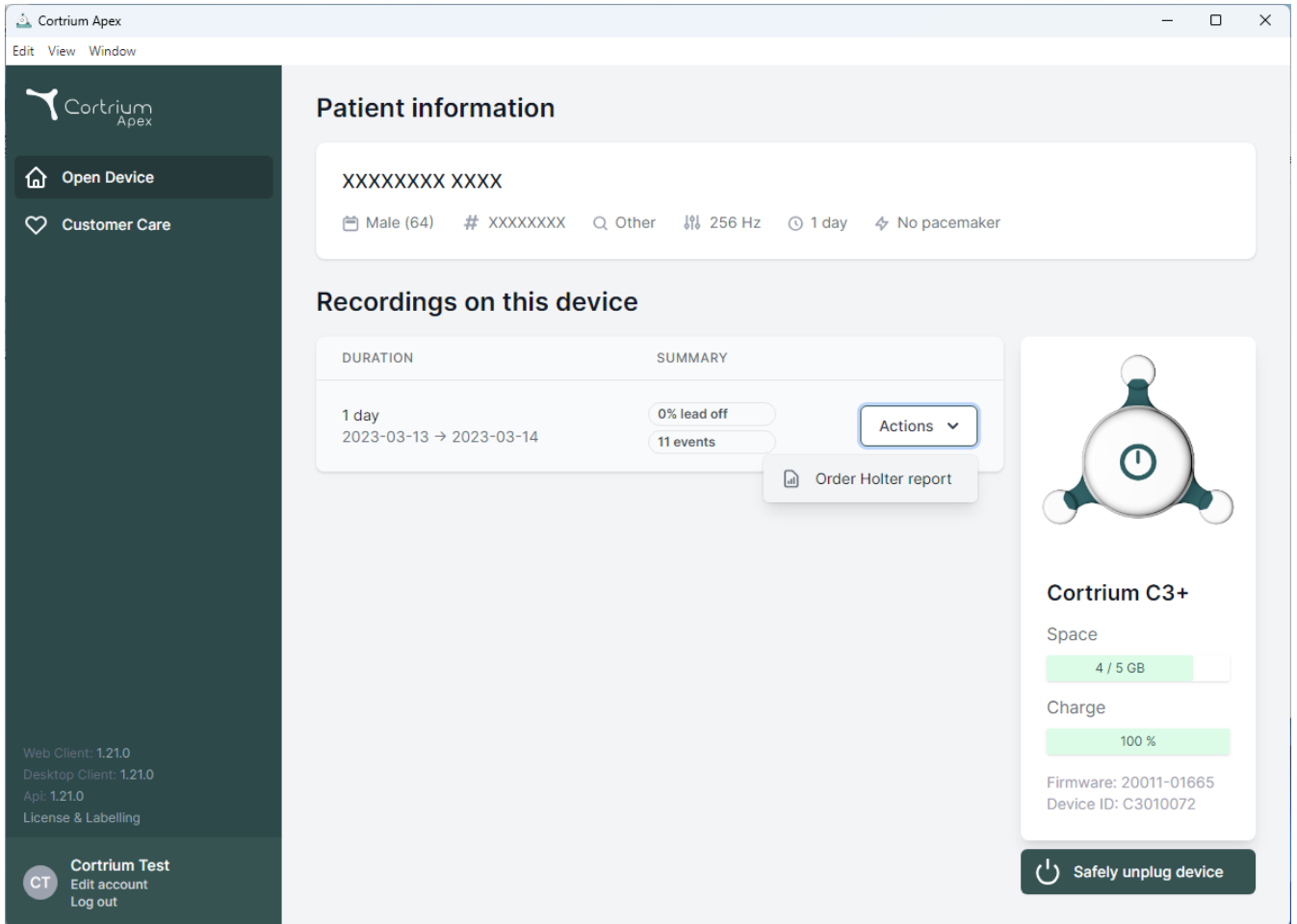
Recording length Pacemaker Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', 'Customer Care', and version information (Web Client: 1.21.0, Desktop Client: 1.21.0, Api: 1.21.0, License & Labelling). The main content area is divided into sections: 'Patient information' showing a masked patient ID (XXXXXXXX XXXX) and details like 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'; 'Recordings on this device' with a table of recordings (1 day, 2023-03-13 → 2023-03-14) and summary statistics (0% lead off, 11 events); and a device status panel for 'Cortrium C3+' showing 'Space' (4 / 5 GB), 'Charge' (100%), 'Firmware: 20011-01665', 'Device ID: C3010072', and a 'Safely unplug device' button. An 'Actions' dropdown menu is visible next to the recording summary, with 'Order Holter report' as an option.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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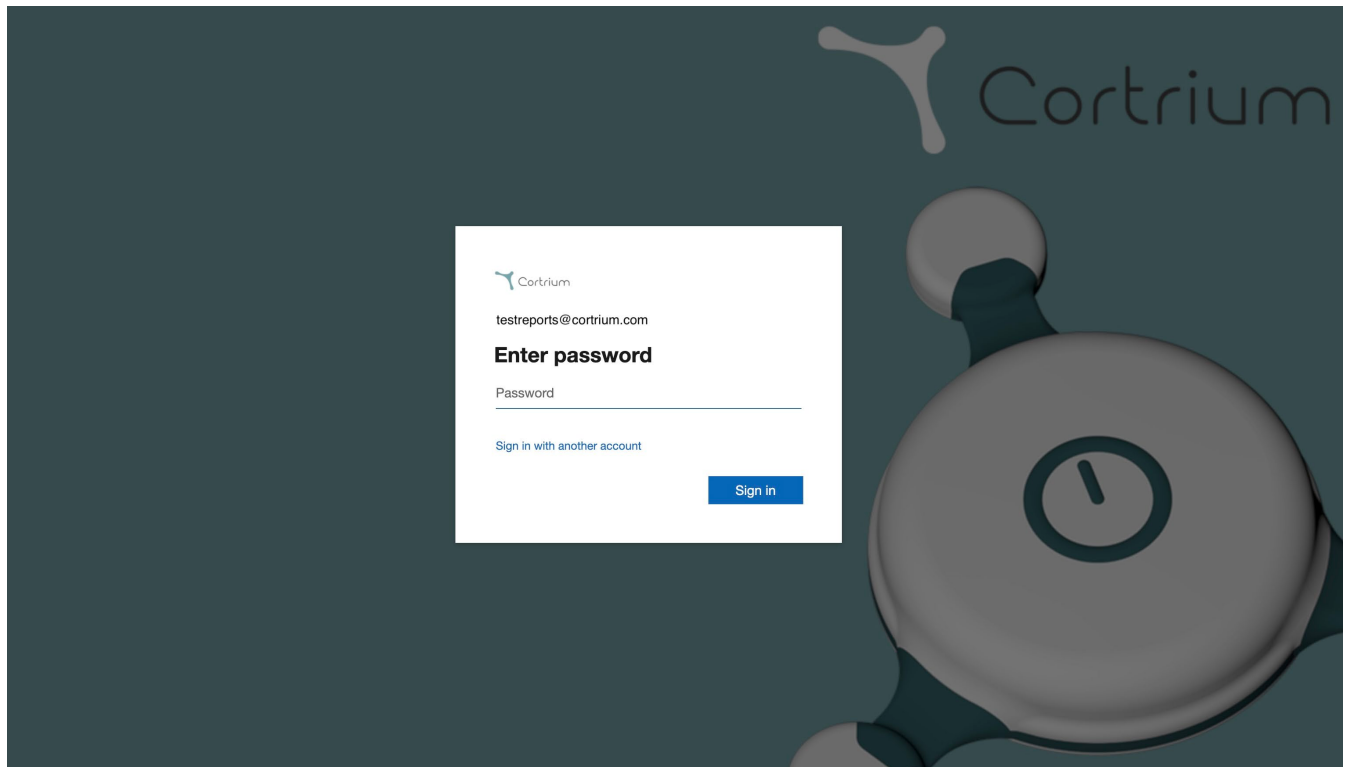
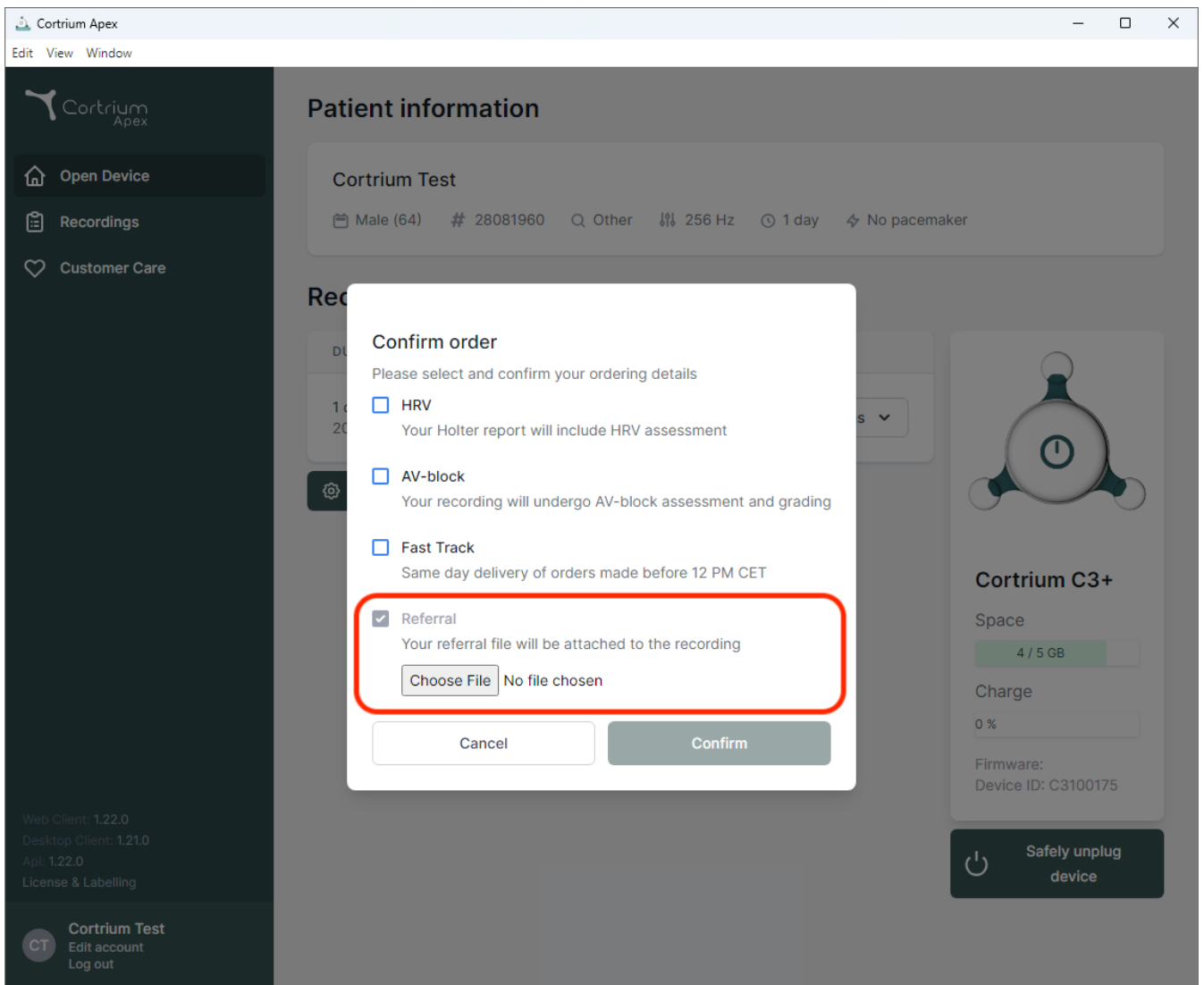
The screenshot shows the Cortrium Apex web application interface. A 'Confirm order' dialog box is displayed in the center, overlaid on the 'Patient information' page. The dialog box contains the following options:

- ☐ HRV
Your Holter report will include HRV assessment
- ☐ AV-block
Your recording will undergo AV-block assessment and grading
- ☐ Fast Track
Same day delivery of orders made before 12 PM CET
- ☒ Referral
Your referral file will be attached to the recording

The 'Referral' option is highlighted with a red rectangle. Below the 'Referral' option, there is a 'Choose File' button and the text 'No file chosen'. At the bottom of the dialog box, there are 'Cancel' and 'Confirm' buttons.

The background interface shows the 'Patient information' section with details for 'Cortrium Test' (Male (64), # 28081960, Q Other, 256 Hz, 1 day, No pacemaker). On the right, there is a section for 'Cortrium C3+' showing 'Space' (4 / 5 GB), 'Charge' (0 %), and 'Firmware: Device ID: C3100175'. A 'Safely unplug device' button is visible at the bottom right.

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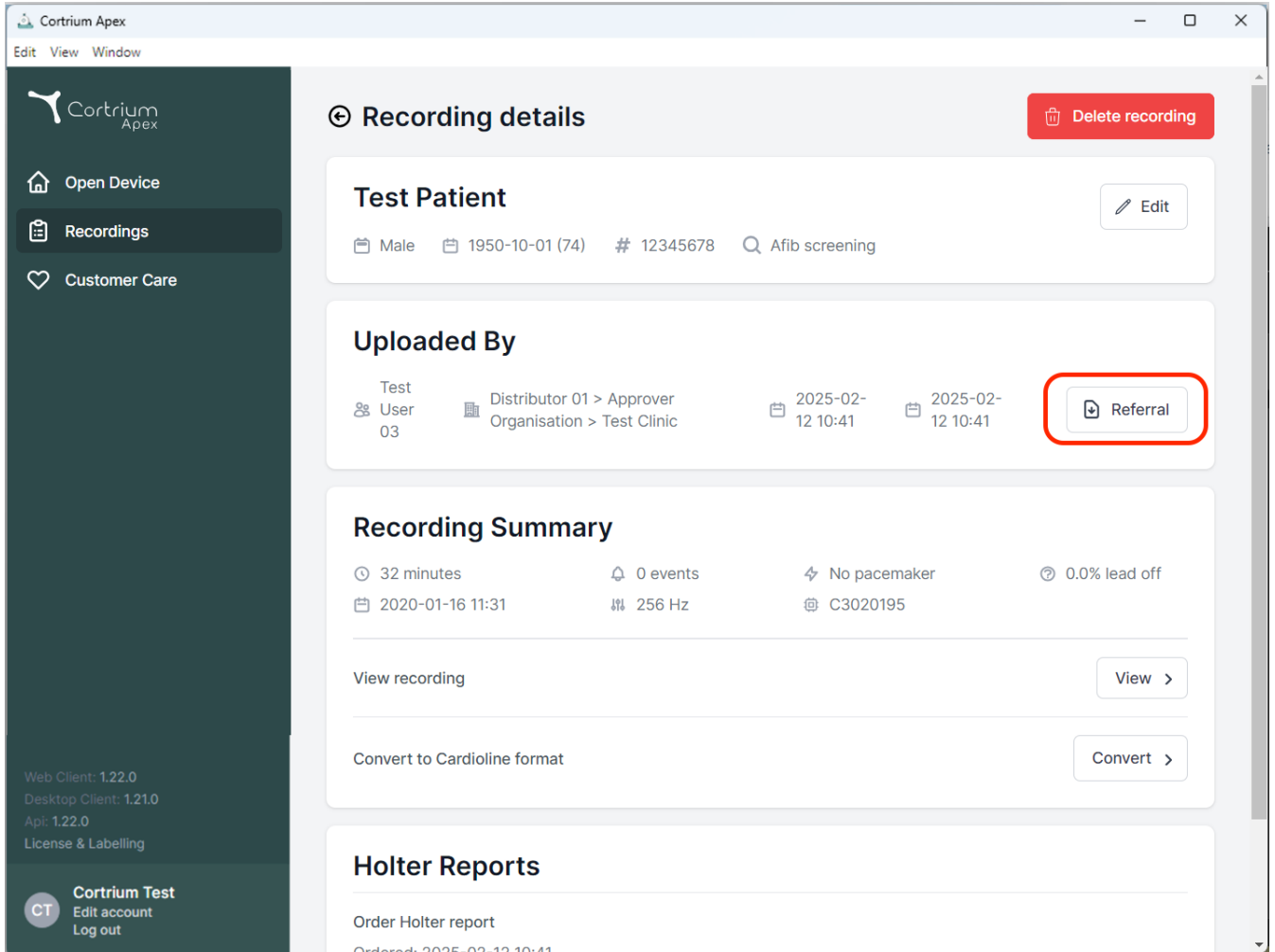The screenshot displays the 'Recording details' page in the Cortrium Apex web application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right corner. Below this, the 'Test Patient' section shows patient information: Male, 1950-10-01 (74), ID # 12345678, and Afib screening. An 'Edit' button is available. The 'Uploaded By' section lists the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and upload dates (2025-02-12 10:41). A 'Referral' button is highlighted with a red box. The 'Recording Summary' section provides details: 32 minutes duration, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It also shows a frequency of 256 Hz and a device ID C3020195. At the bottom, there are buttons for 'View recording', 'Convert to Cardioline format', and 'View' and 'Convert' buttons. The 'Holter Reports' section is partially visible at the bottom.

Cortrium Apex

Edit View Window

Recording details Delete recording

Test Patient Edit

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off

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
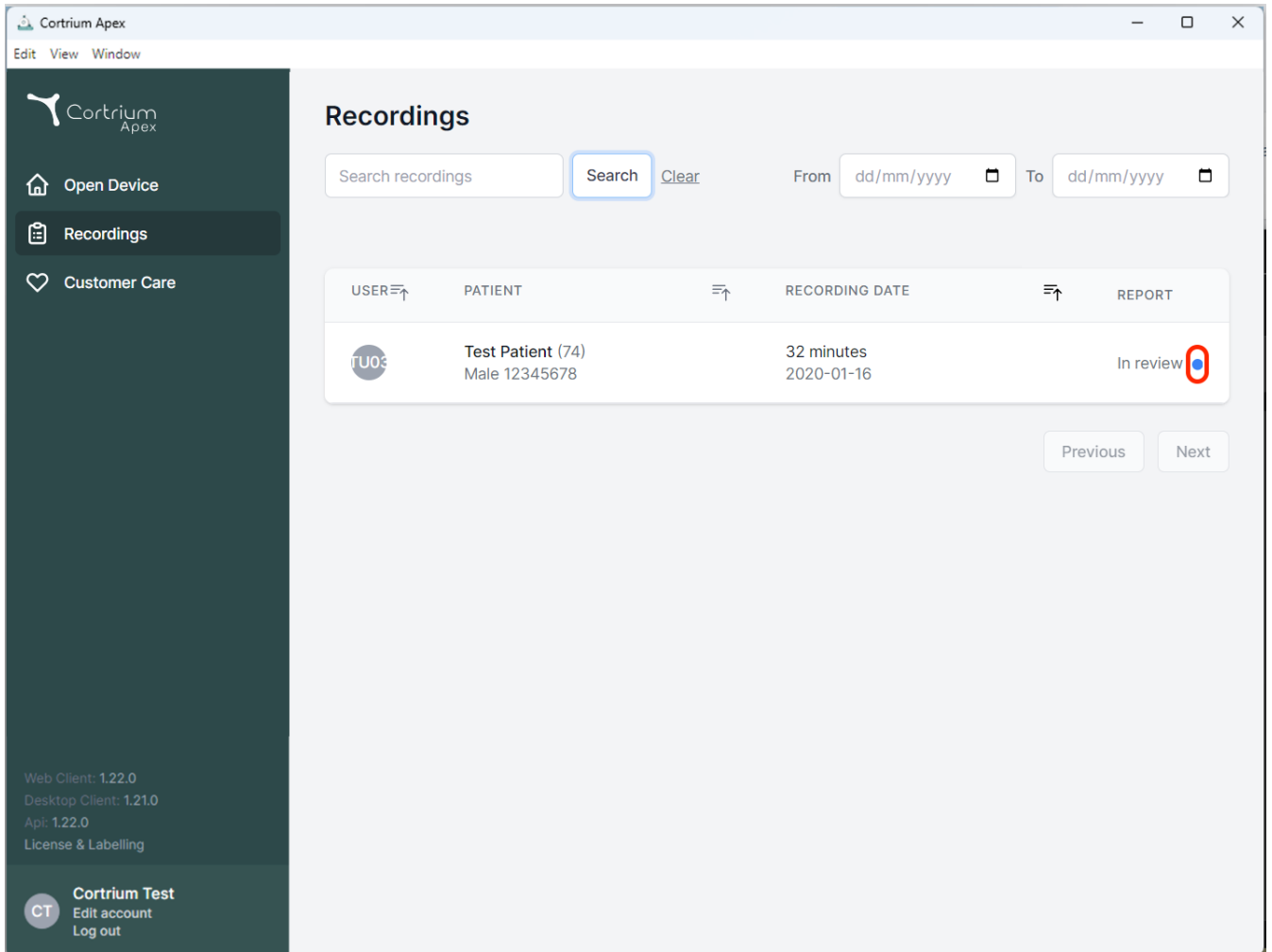
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot icon is visible next to the "In review" status, which is circled in red. The bottom of the sidebar shows version information and a "Cortrium Test" button.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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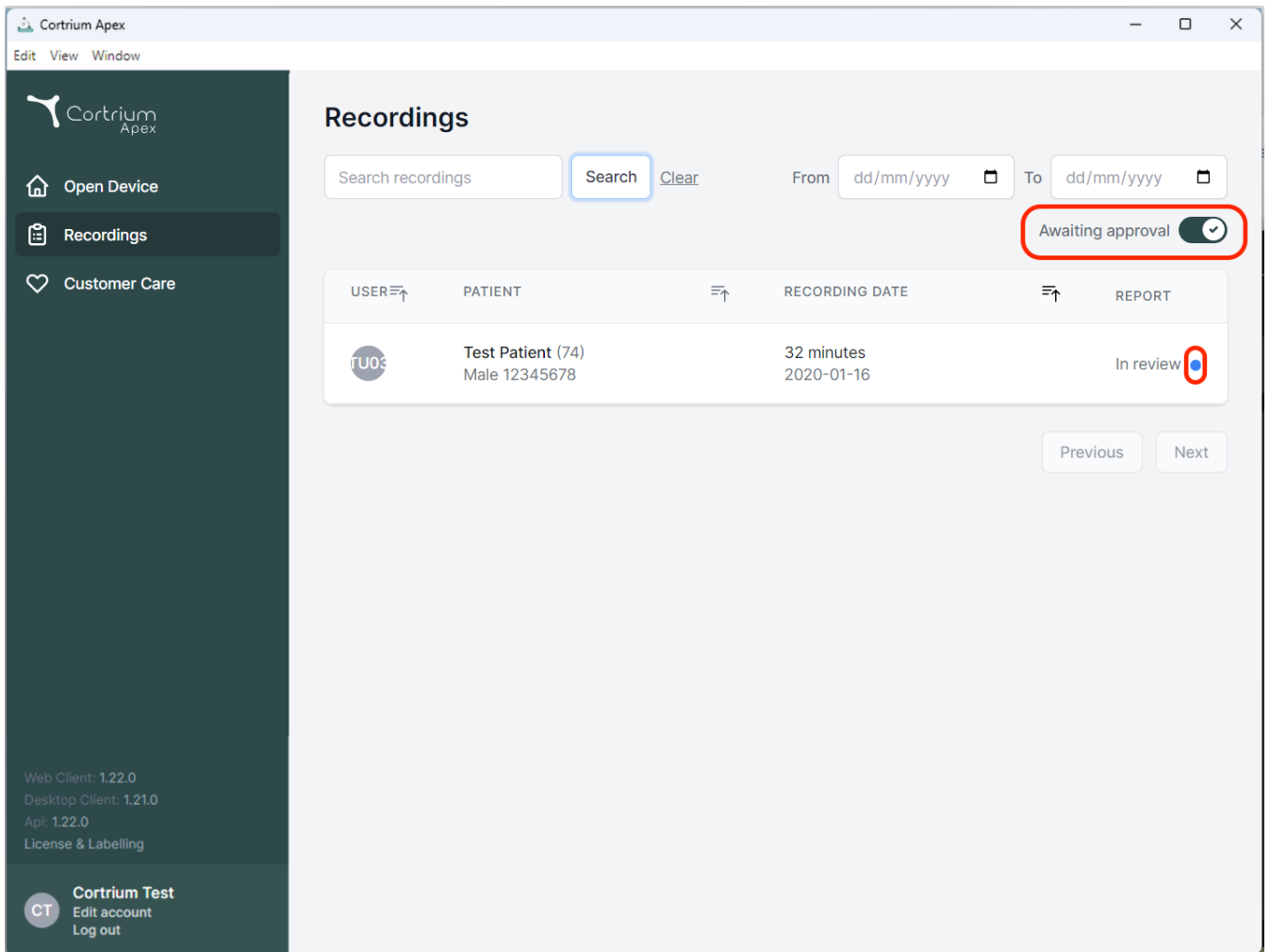From To

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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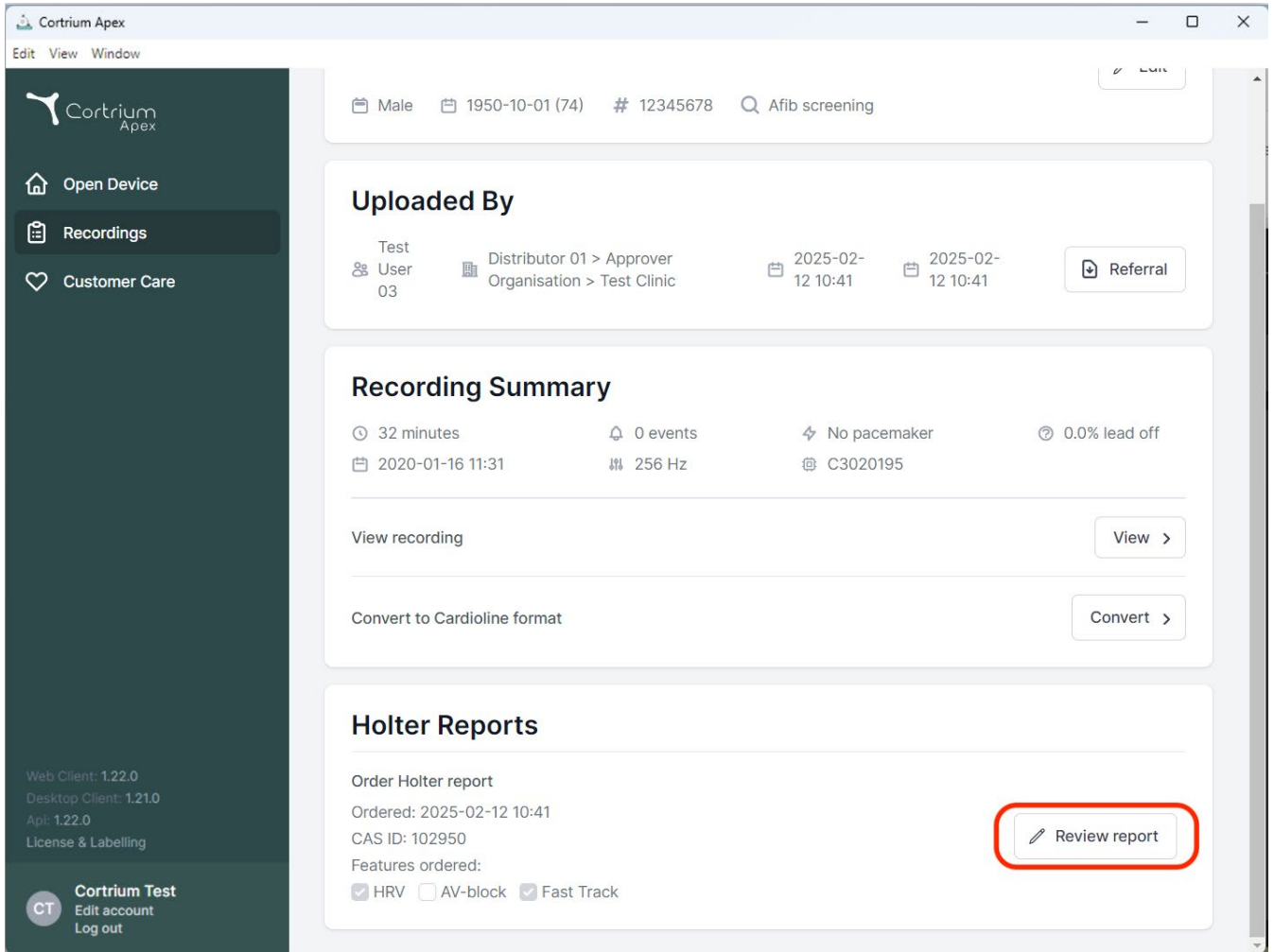The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and three main menu items: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. Below these, version information is listed: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes 'Edit account' and 'Log out' links.

The main content area shows patient information at the top: 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'. Below this, the 'Uploaded By' section lists 'Test User 03' as the distributor, with approval from 'Distributor 01 > Approver Organisation > Test Clinic' on '2025-02-12 10:41'. A 'Referral' button is present.

The 'Recording Summary' section provides details about the recording: '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off'. It also shows the recording date '2020-01-16 11:31', a frequency of '256 Hz', and a device ID 'C3020195'. Action buttons for 'View recording' and 'Convert to Cardioline format' are available.

The 'Holter Reports' section shows an 'Order Holter report' with an order date of '2025-02-12 10:41' and a CAS ID of '102950'. It lists ordered features: 'HRV' (checked), 'AV-block' (unchecked), and 'Fast Track' (checked). A 'Review report' button, highlighted with a red rectangle, is located in the bottom right corner of this section.

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