# Cortrium Apex 1.22

## Mode d'emploi

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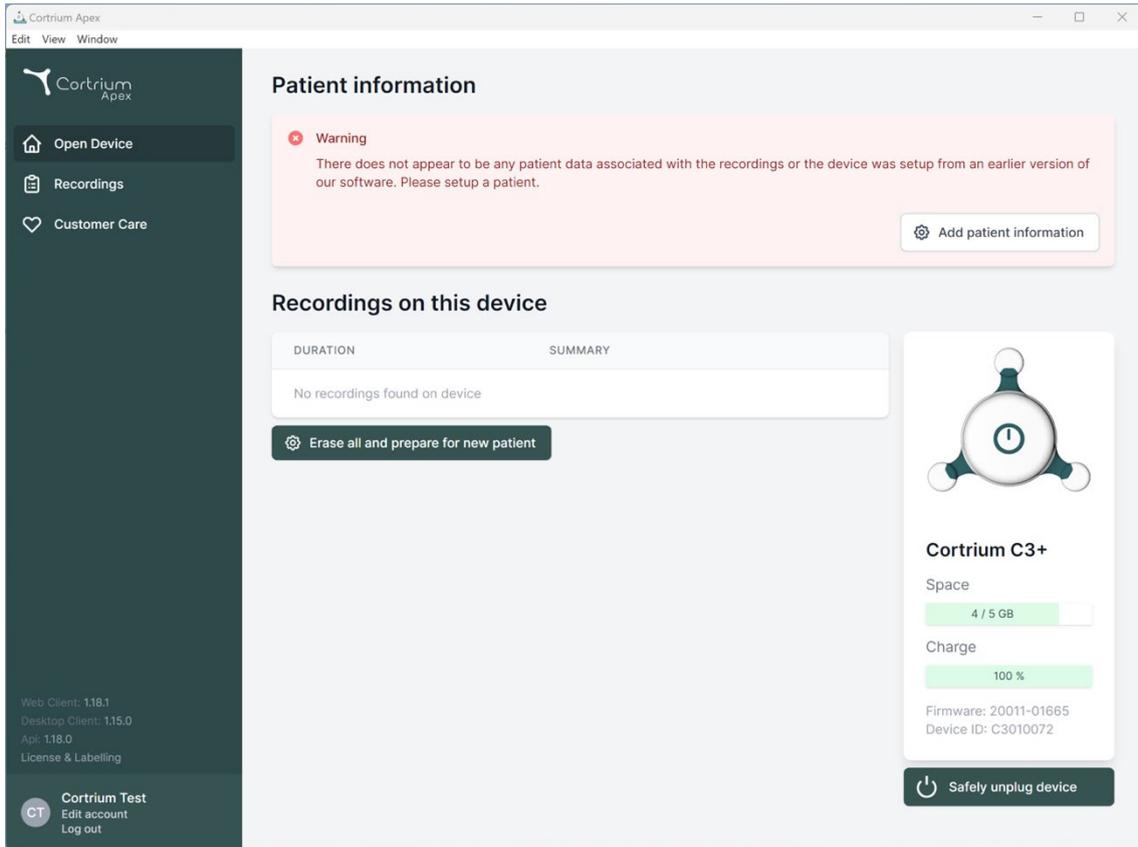
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client (1.18.1), Desktop Client (1.15.0), and App (1.18.0), along with a 'License & Labelling' link. At the bottom of the sidebar is a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' An 'Add patient information' button is located to the right of the warning.

Below the warning is a section titled 'Recordings on this device' which contains a table with columns 'DURATION' and 'SUMMARY'. The table is currently empty, displaying 'No recordings found on device'. A button labeled 'Erase all and prepare for new patient' is positioned below the table.

On the right side of the main area, there is a device status card for 'Cortrium C3+'. It includes a device icon, the model name, and two progress indicators: 'Space' at 4 / 5 GB and 'Charge' at 100%. Below these are the 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings
- Customer Care

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**

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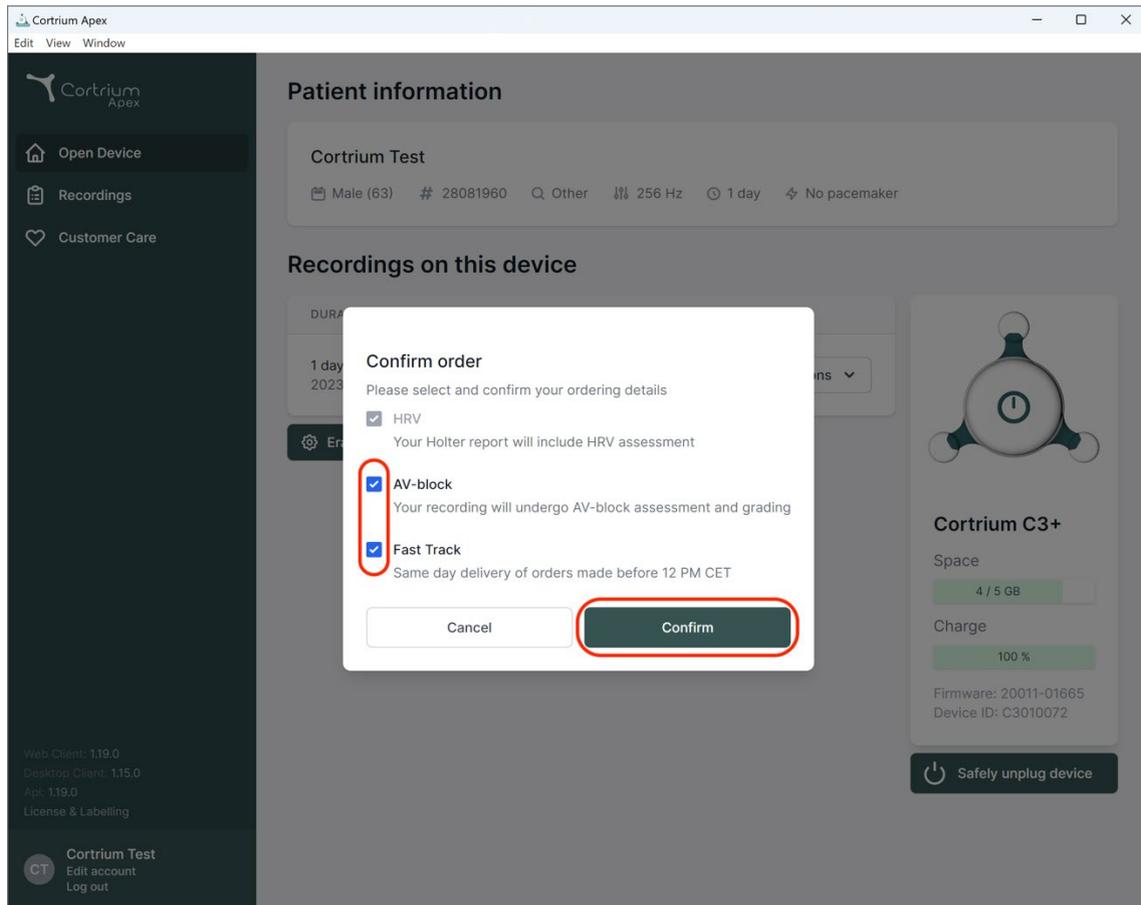
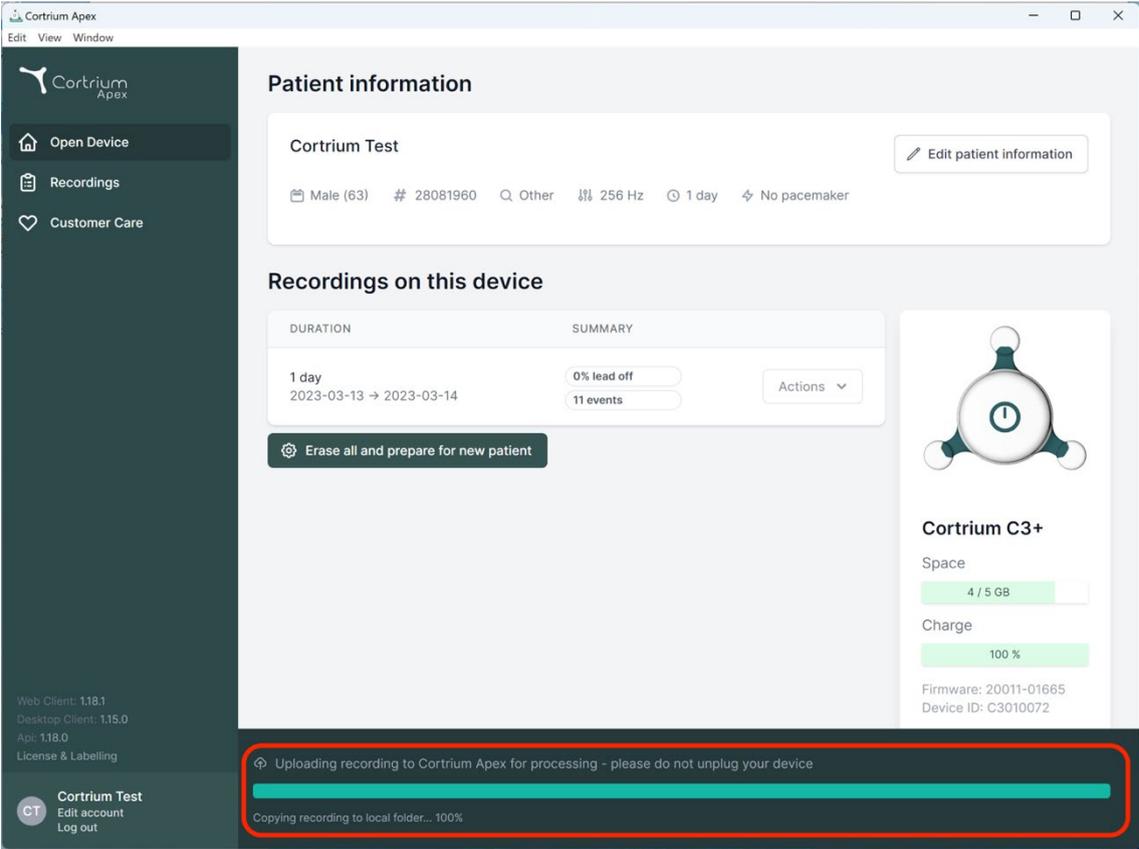
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a summary of 0% lead off and 11 events. A 'Cortrium C3+' device status card on the right indicates 4/5 GB space used and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The sidebar also contains version information and user account options.

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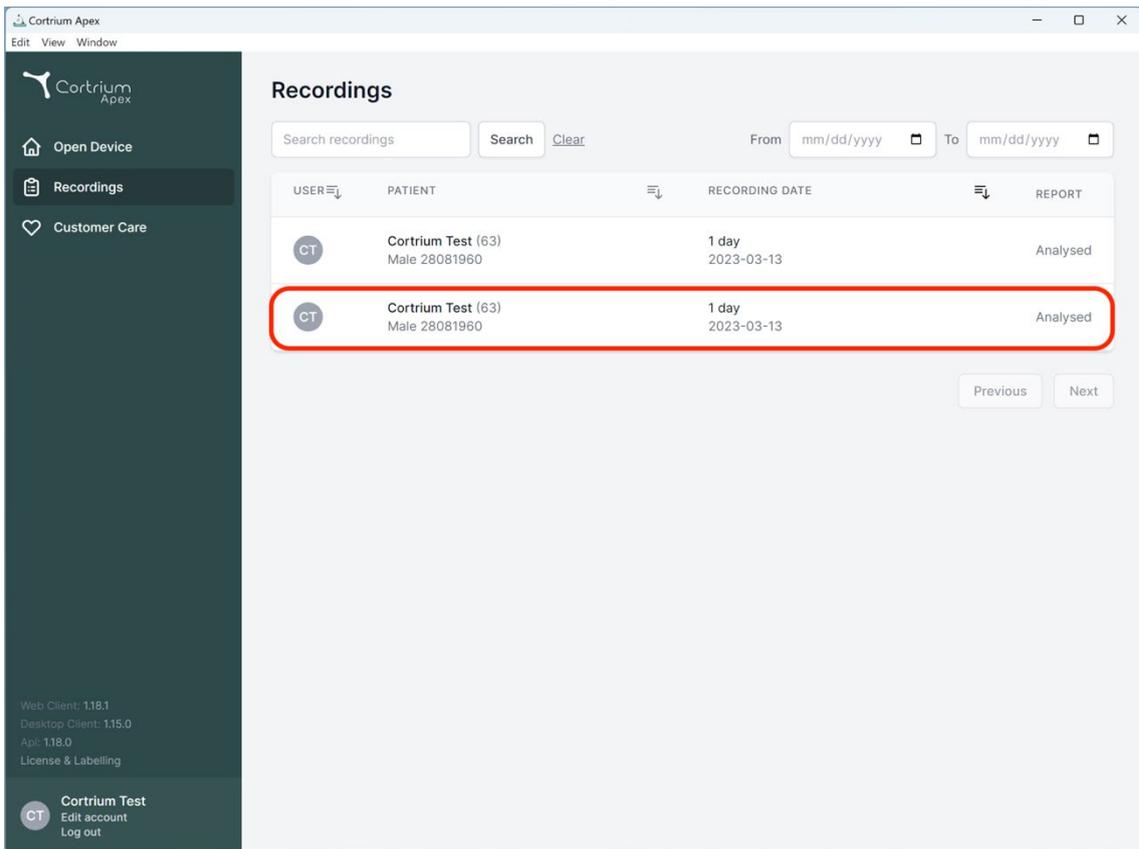
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings (selected), and Customer Care. At the bottom of the sidebar, it shows the user's name 'Cortrium Test' and options to 'Edit account' and 'Log out'. The main content area is titled 'Recordings' and features a search bar, a 'Search' button, a 'Clear' button, and date filters for 'From' and 'To'. Below this is a table with the following data:

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At the bottom right of the table, there are 'Previous' and 'Next' navigation buttons. The second row of the table is highlighted with a red border.

Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**CT** Cortrium Test  
Edit account  
Log out

### Recording details

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

### Recording Summary

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

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Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track

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
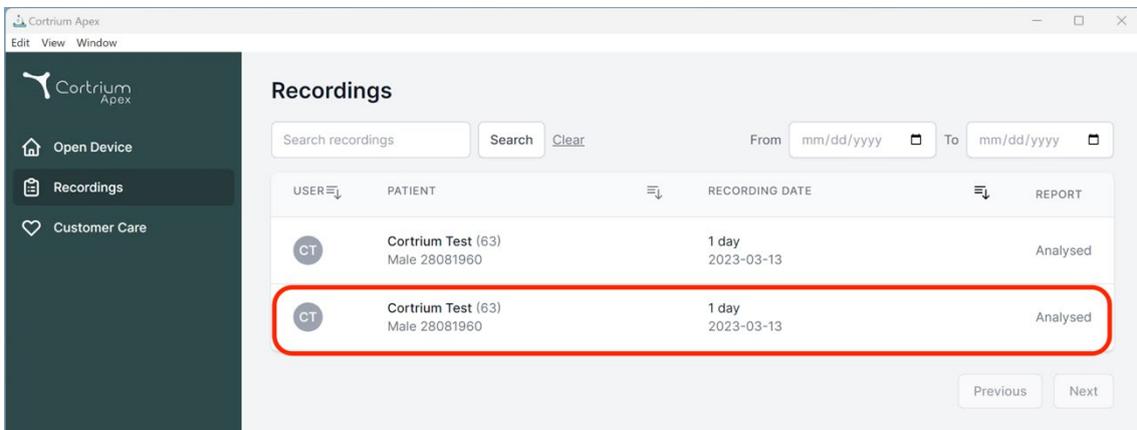
The screenshot shows the 'Recordings' section of the Cortrium Apex software. It features a search bar, a date range selector, and a table with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row of the table is highlighted with a red box.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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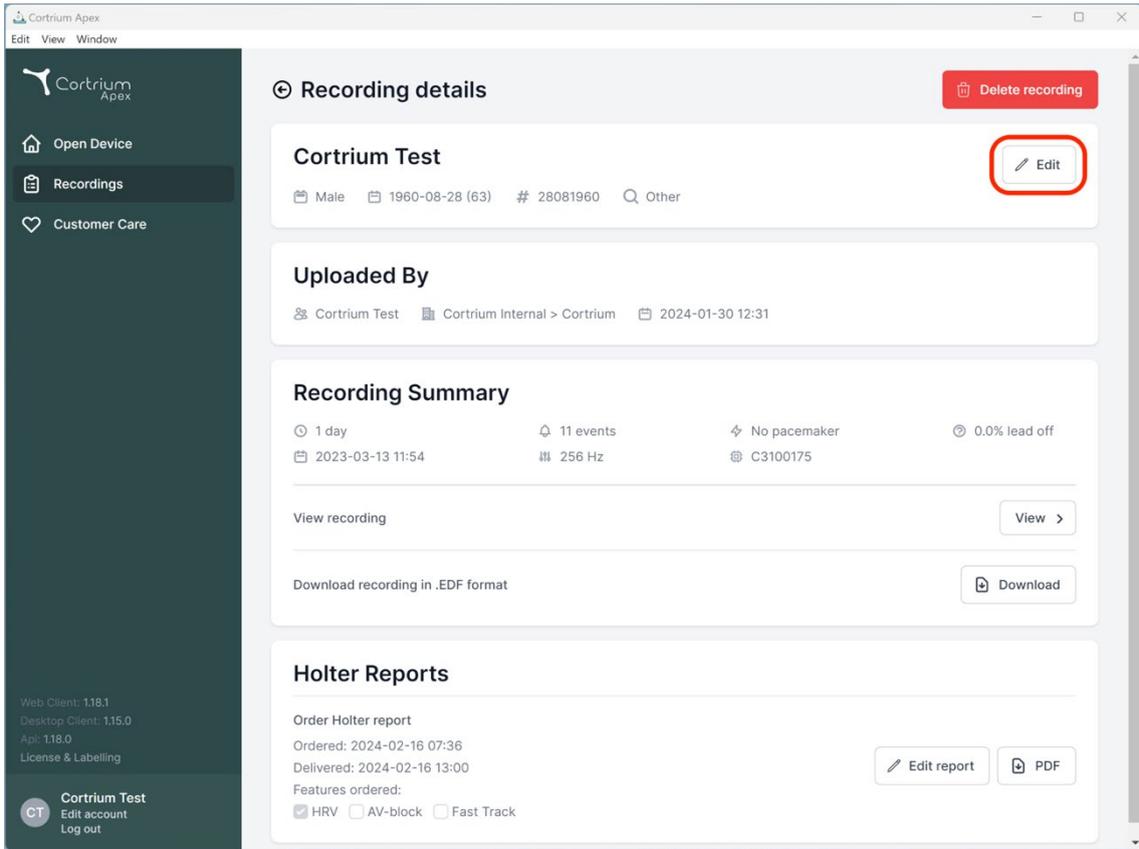
The screenshot shows the 'Recording details' page for a recording titled 'Cortrium Test'. The interface includes a sidebar with navigation options like 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Recording details:** Shows patient information (Male, 1960-08-28, # 28081960) and a red 'Delete recording' button. An 'Edit' button is circled in red.
- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-01-30 12:31'.
- Recording Summary:** Displays key metrics: 1 day duration, 11 events, No pacemaker, 0.0% lead off, and a frequency of 256 Hz. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows an 'Order Holter report' with dates and a 'Features ordered' section where 'HRV' is checked. 'Edit report' and 'PDF' buttons are present.

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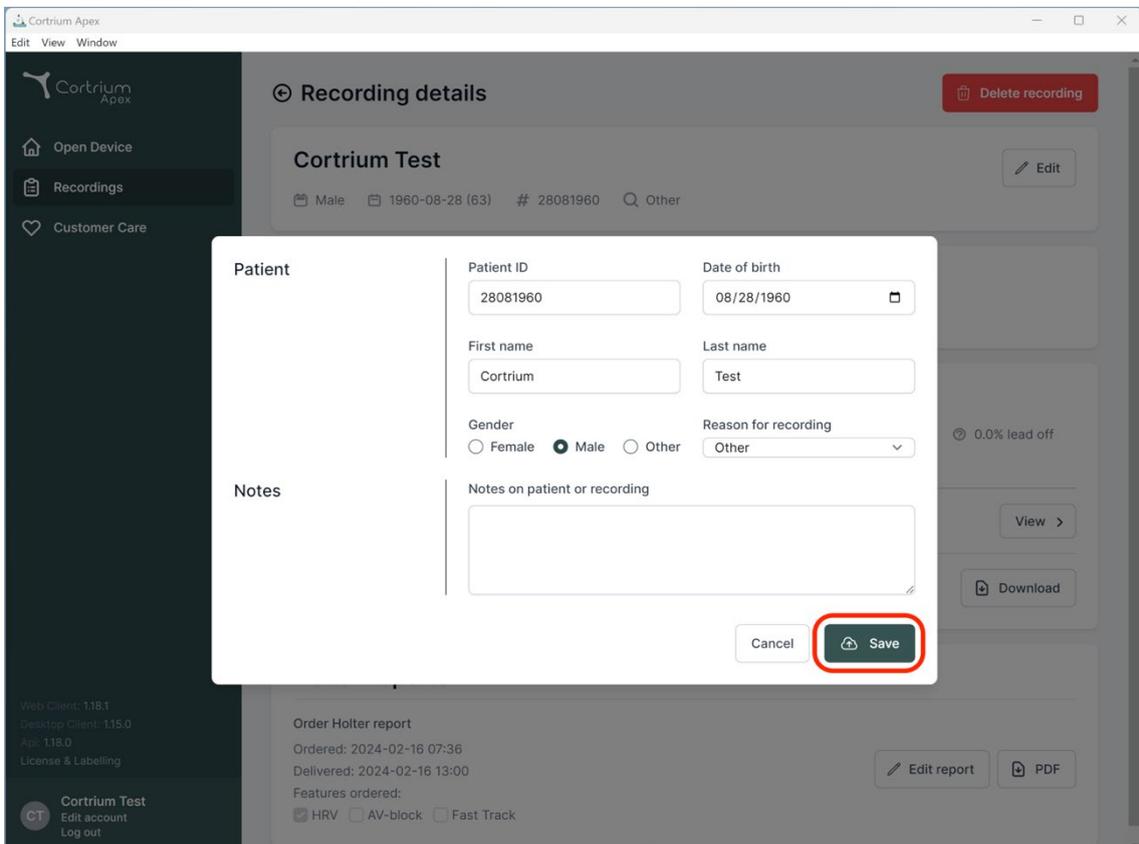This screenshot shows the same 'Recording details' page as above, but with a modal window open for editing patient information. The modal is titled 'Patient' and contains the following fields:


- Patient ID:** 28081960
- Date of birth:** 08/28/1960
- First name:** Cortrium
- Last name:** Test
- Gender:** Radio buttons for Female, Male (selected), and Other.
- Reason for recording:** A dropdown menu currently set to 'Other'.
- Notes:** A text area labeled 'Notes on patient or recording'.

At the bottom of the modal, there are 'Cancel' and 'Save' buttons. The 'Save' button is circled in red, indicating the action to be taken to save the changes.

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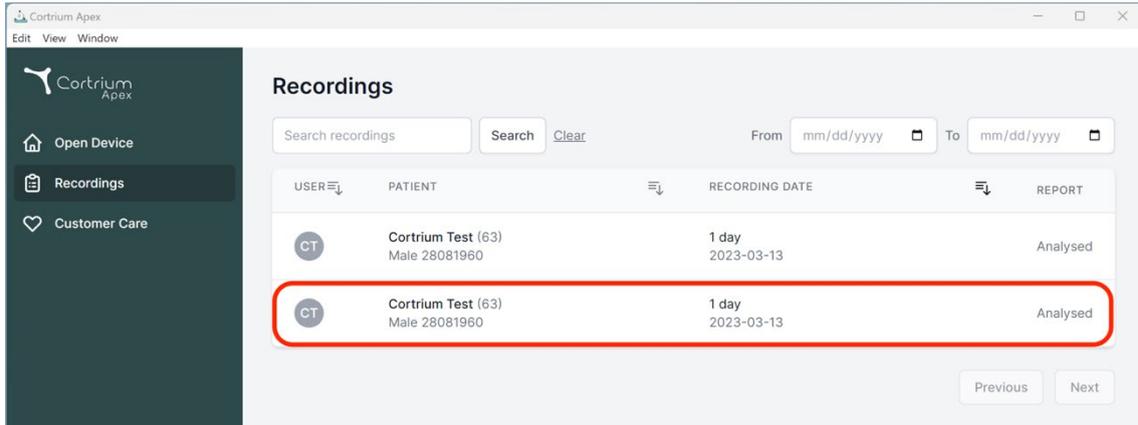
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. It features a search bar and a table with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row of the table is highlighted with a red border.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

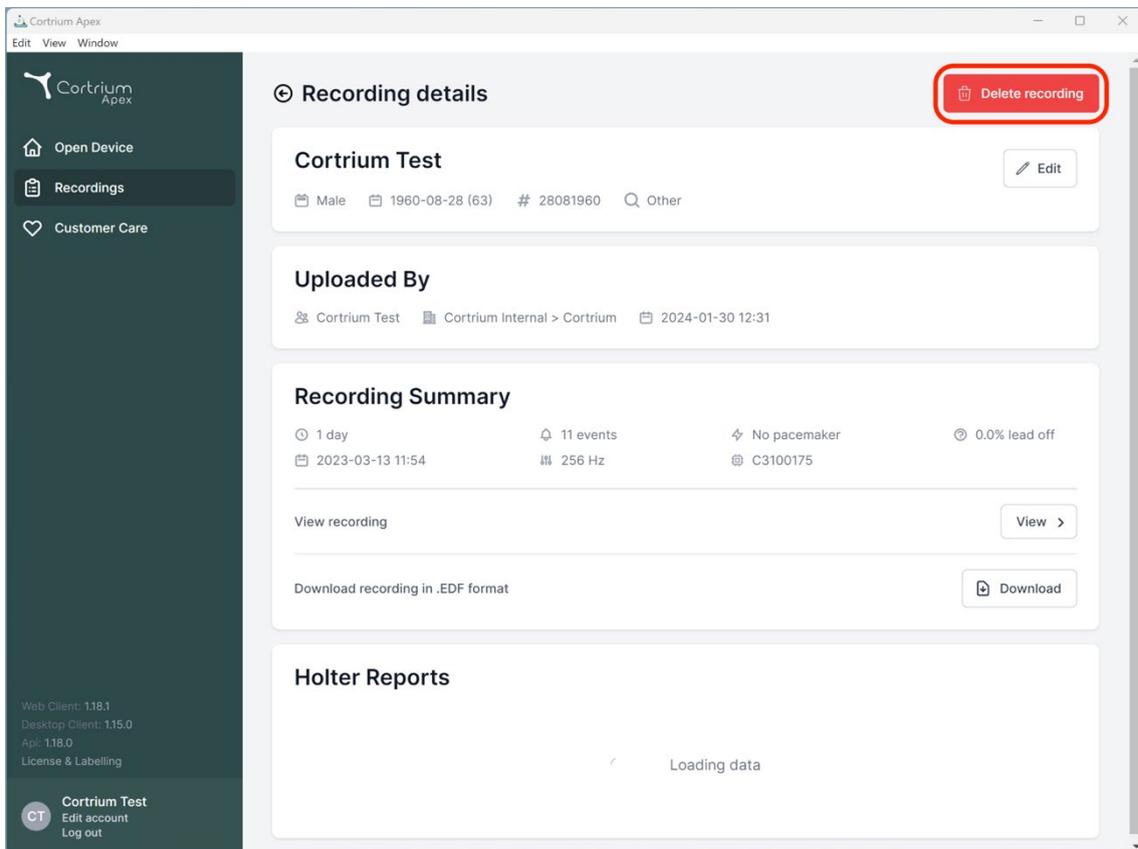The screenshot shows the 'Recording details' page for 'Cortrium Test'. A 'Delete recording' button is highlighted with a red box. The page displays patient information, upload details, recording summary, and options to view or download the recording.

**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**  
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

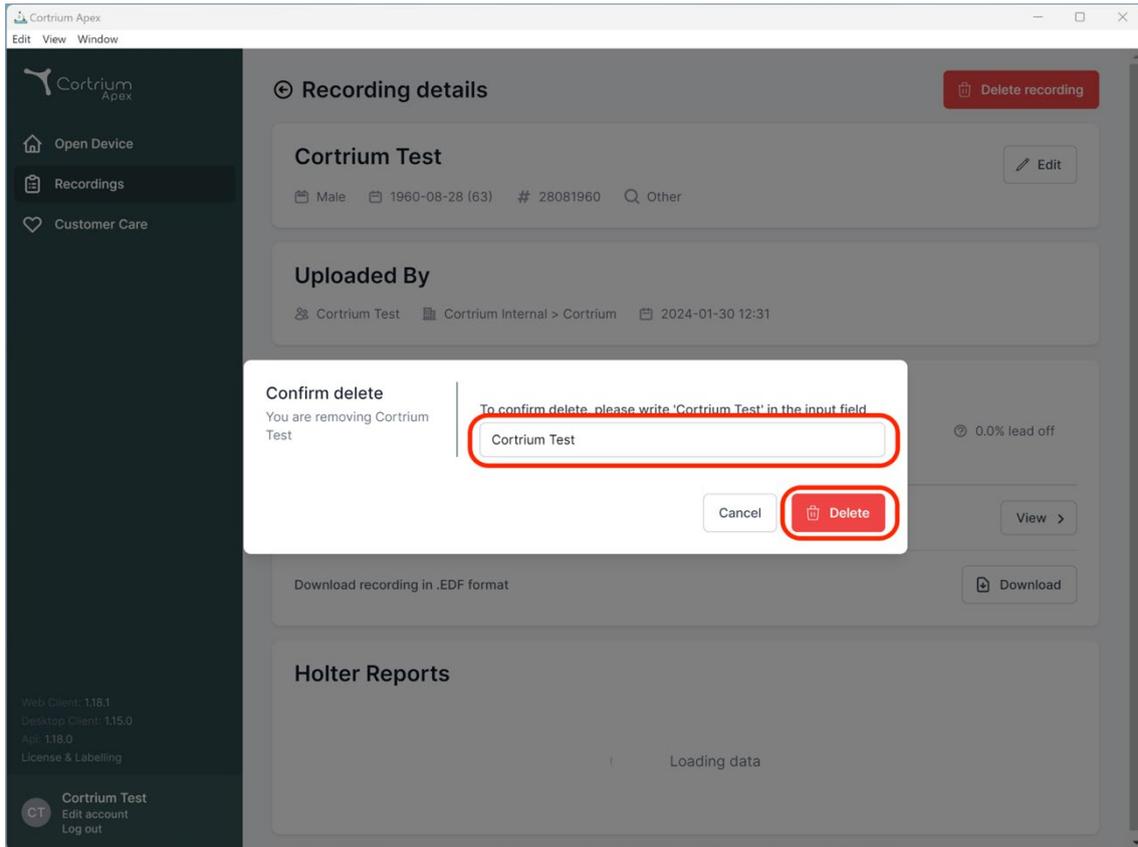
- 1 day
- 11 events
- No pacemaker
- 0.0% lead off
- 2023-03-13 11:54
- 256 Hz
- C3100175

View recording View >

Download recording in .EDF format Download

**Holter Reports**

Loading data

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

### Recording details

Delete recording

#### Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Edit

#### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

0.0% lead off

View >

Download recording in .EDF format

Download

#### Holter Reports

Loading data

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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
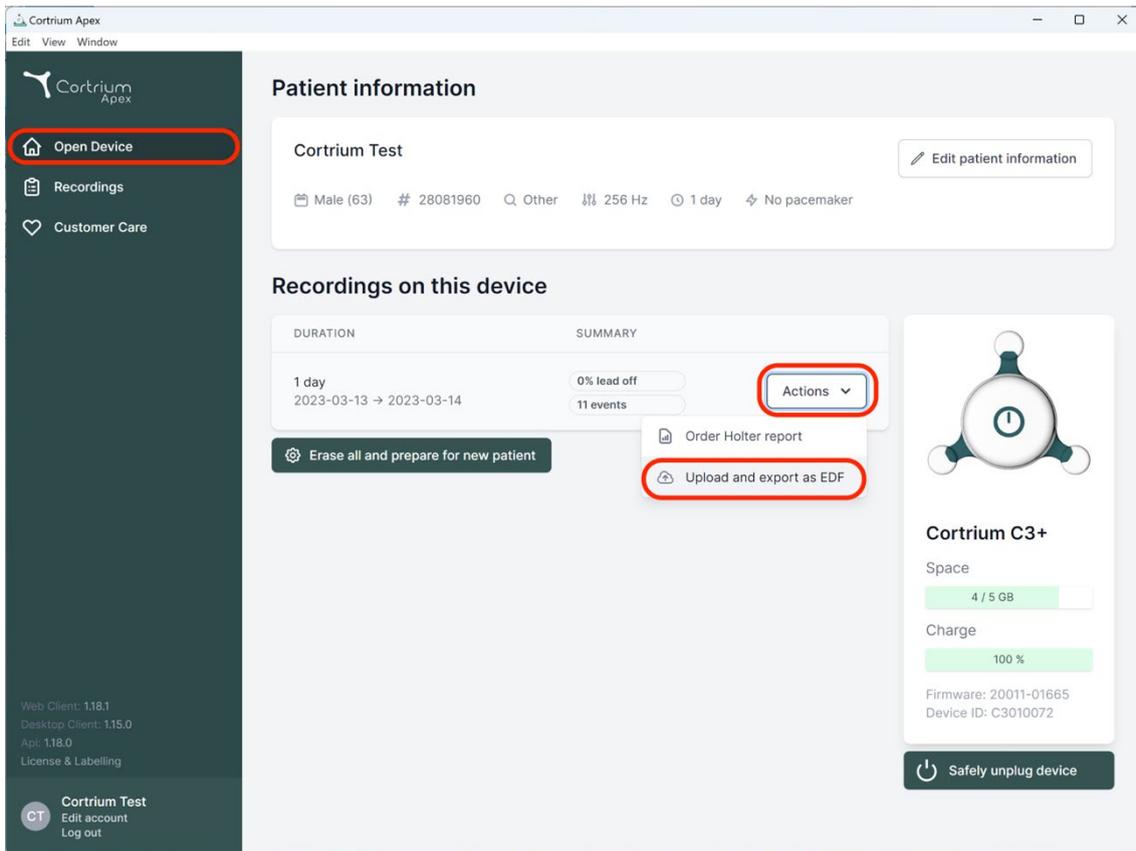
The screenshot displays the Cortrium Apex web application interface. On the left, a dark sidebar contains navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and device settings. Below this, the 'Recordings on this device' section shows a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown menu (highlighted with a red box) is open, showing options like 'Order Holter report' and 'Upload and export as EDF' (highlighted with a red box). On the right, a 'Cortrium C3+' device status card shows 'Space: 4 / 5 GB', 'Charge: 100%', and 'Firmware: 20011-01665'. A 'Safely unplug device' button is at the bottom right.

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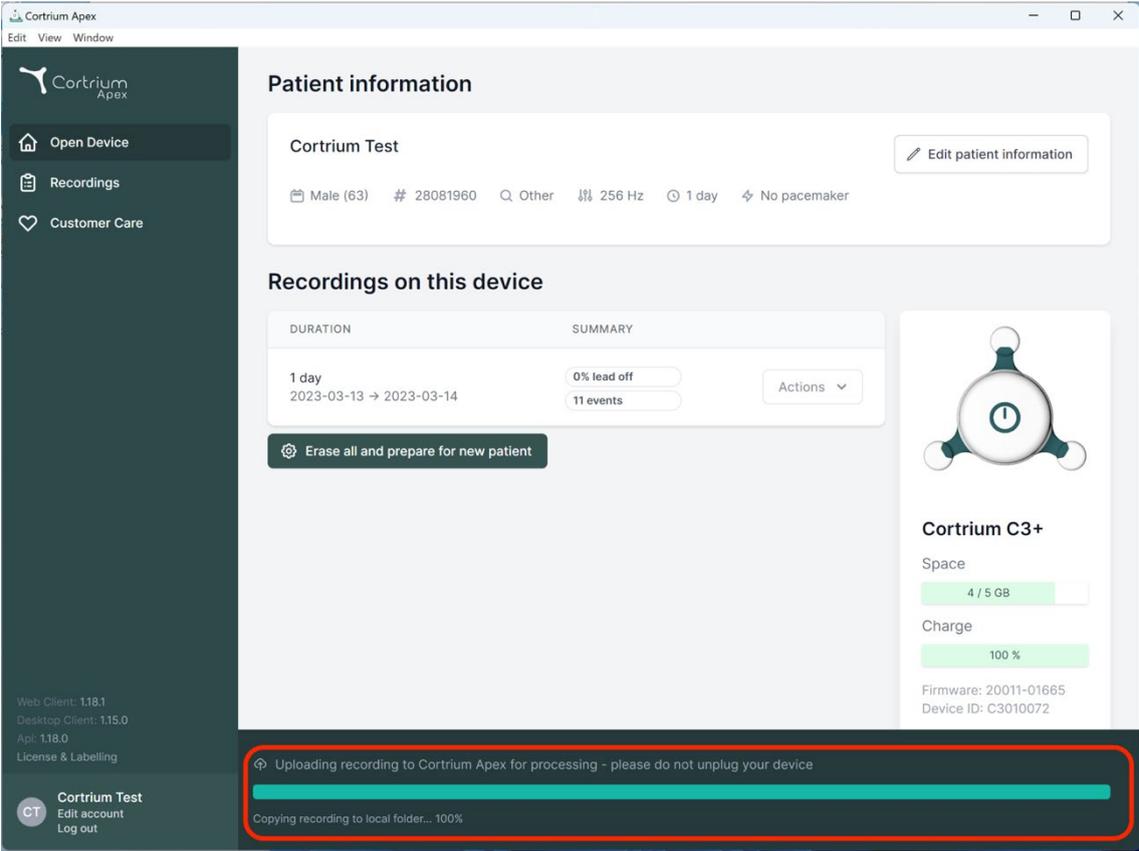Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

Erase all and prepare for new patient

**Cortrium C3+**

Space: 4 / 5 GB

Charge: 100 %

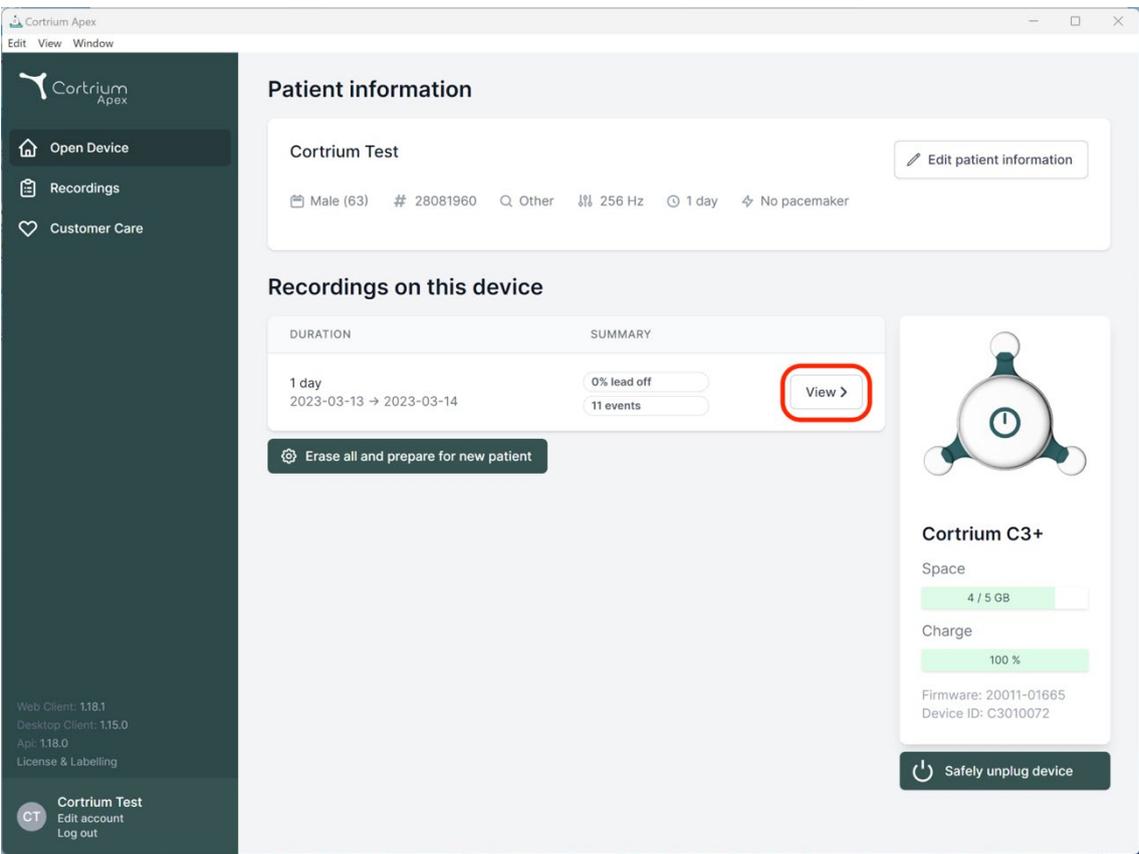
Firmware: 20011-01665  
Device ID: C3010072

Web Client: 1.18.1  
Desktop Client: 1.15.0  
Api: 1.18.0  
License & Labelling

Cortrium Test  
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

**Cortrium Apex**  
Edit View Window

**Patient information**

Cortrium Test Edit patient information

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

Erase all and prepare for new patient

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
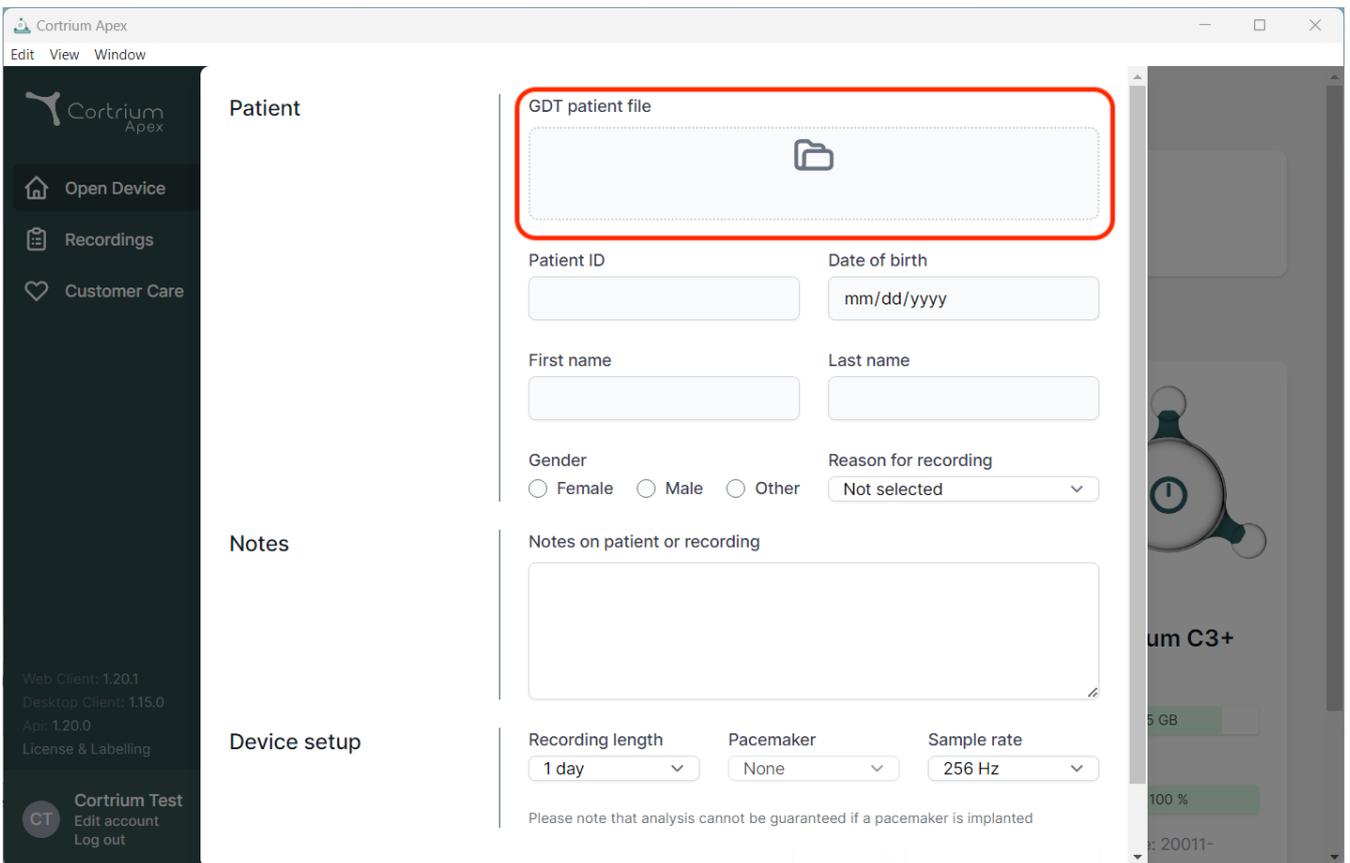
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" field, which is a file upload area with a folder icon. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently set to "Not selected"). There is also a "Notes" section with a text area labeled "Notes on patient or recording". At the bottom, the "Device setup" section includes "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A disclaimer at the bottom of the device setup section states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar shows the user's name "Cortrium Test" and options to "Edit account" and "Log out". The top of the browser window shows the title "Cortrium Apex" and standard window controls.

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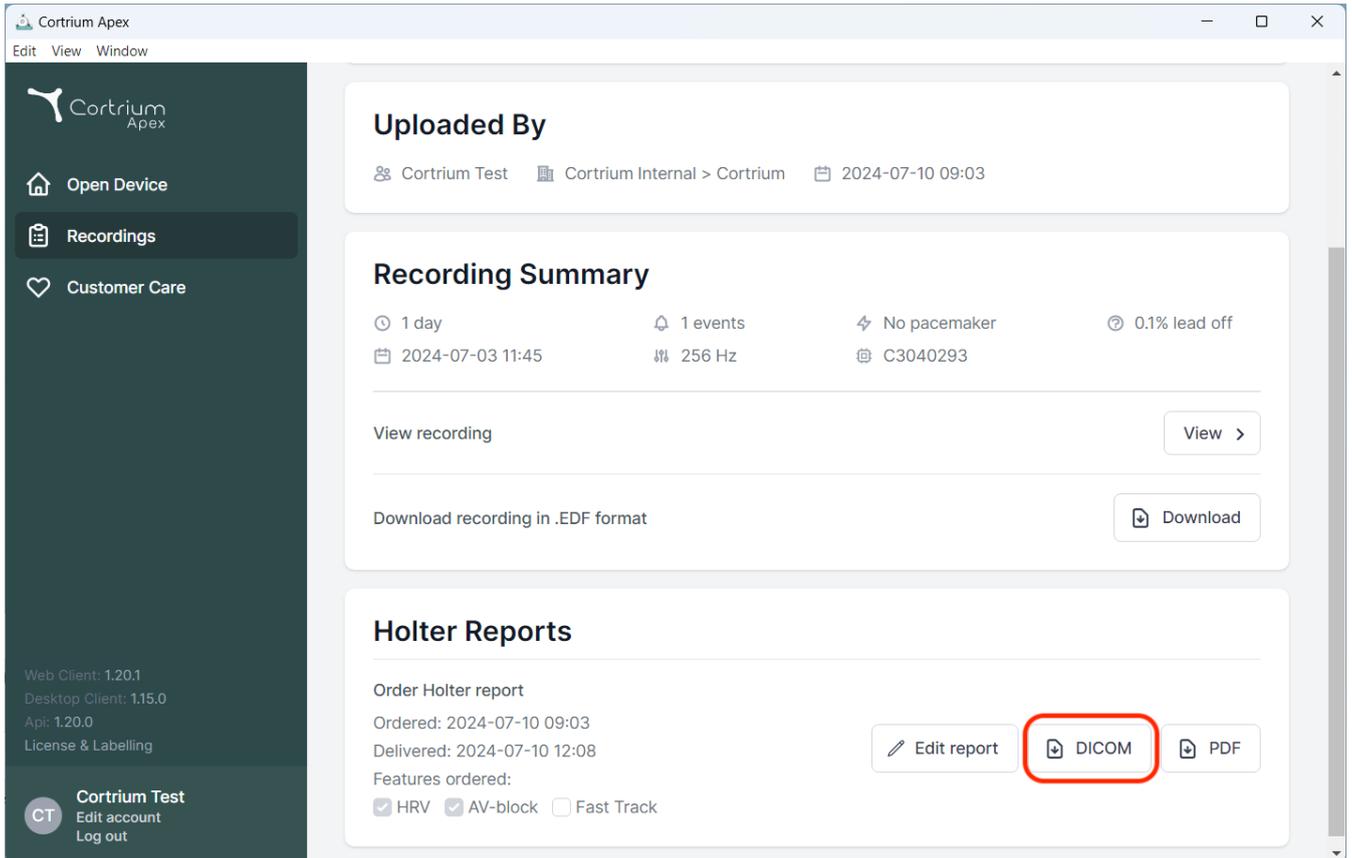
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and 'License & Labelling' are also visible.

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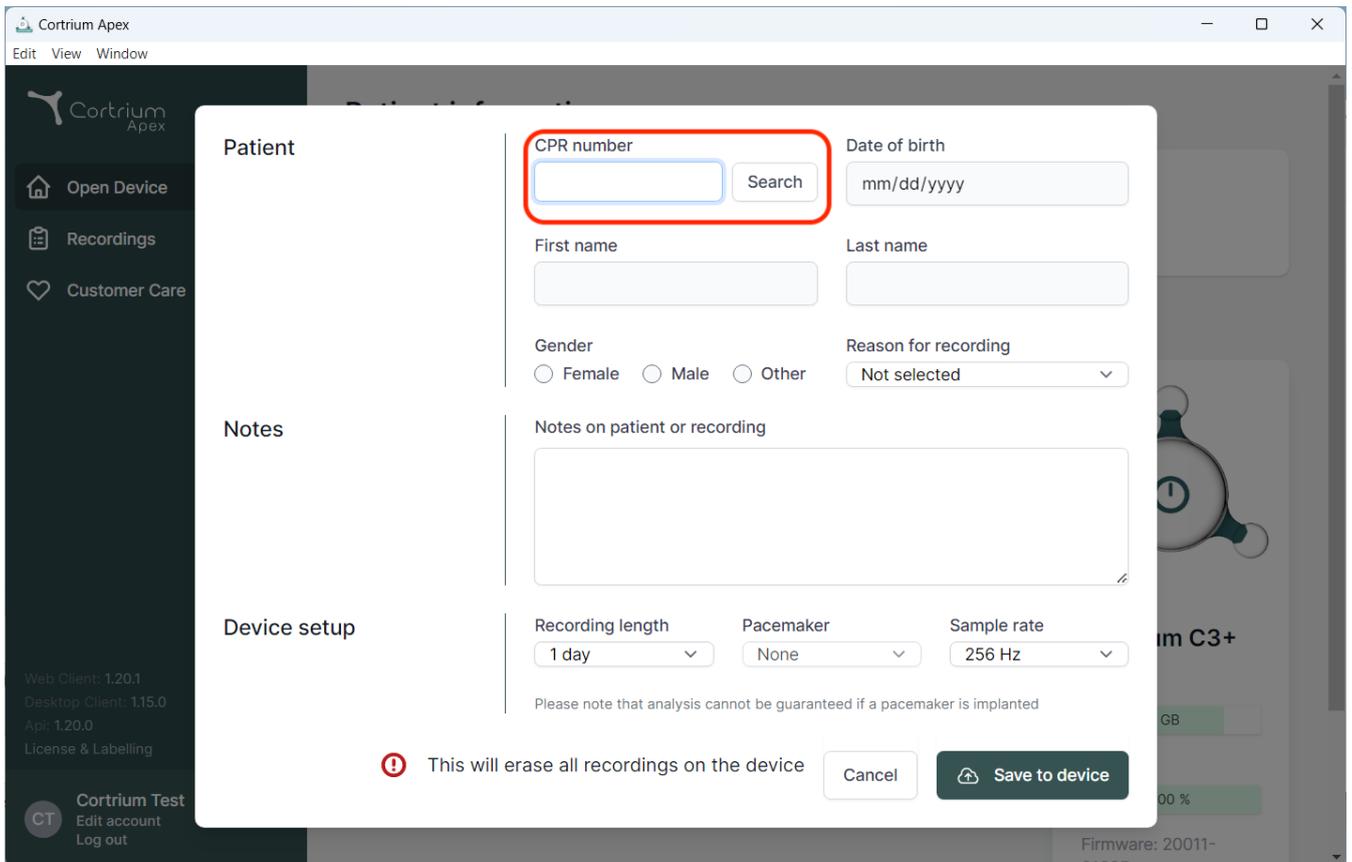
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a search form. The "CPR number" field is highlighted with a red box, and a "Search" button is next to it. The form includes the following fields and options:

- Date of birth:** mm/dd/yyyy
- First name:** [Text input field]
- Last name:** [Text input field]
- Gender:** Radio buttons for Female, Male, and Other.
- Reason for recording:** A dropdown menu currently set to "Not selected".
- Notes:** A large text area for "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the form, a warning message with a red exclamation mark icon states: "This will erase all recordings on the device". At the bottom right of the modal, there are "Cancel" and "Save to device" buttons.

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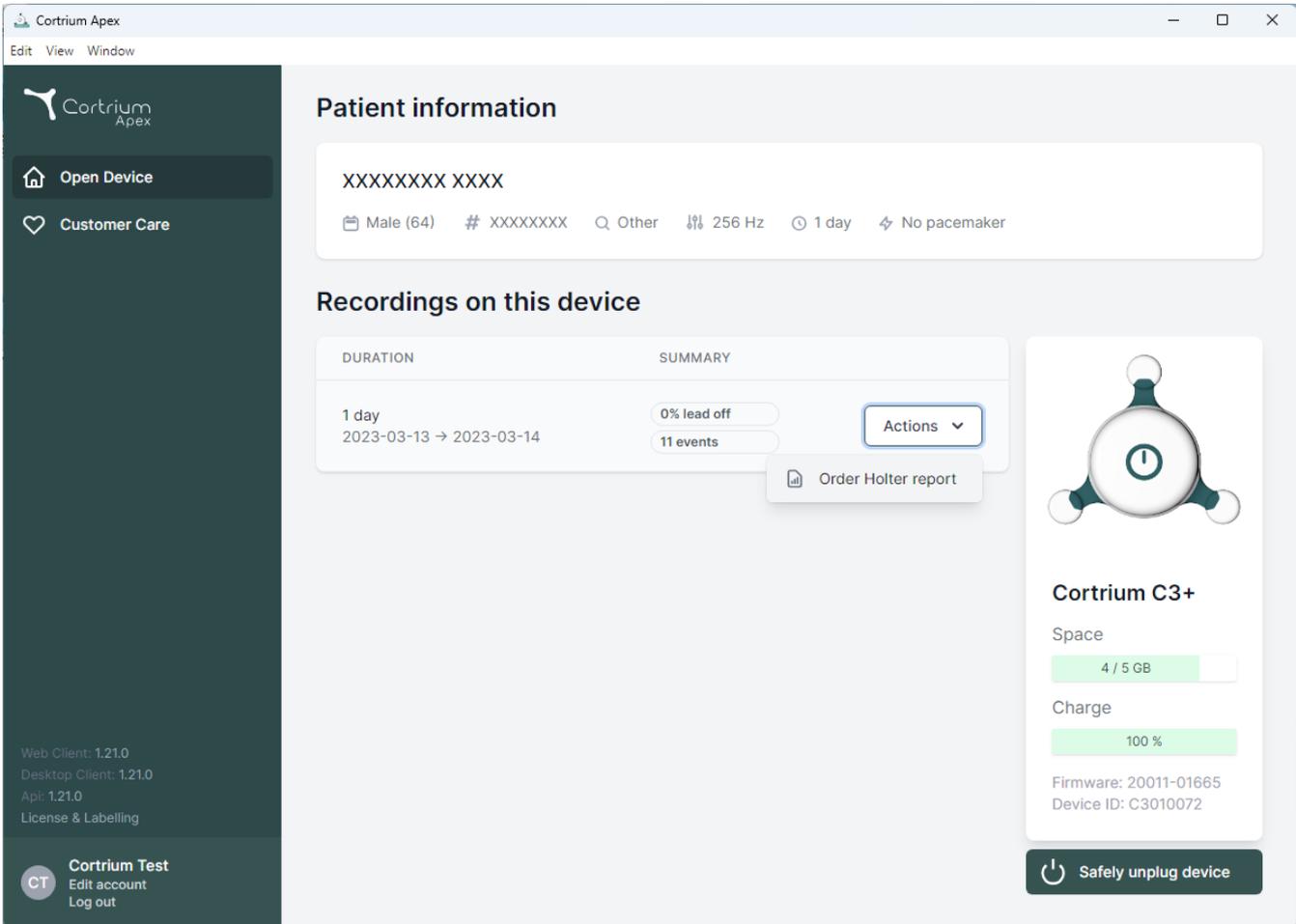
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window'. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' and 'Customer Care'. Below these are version details: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with columns for 'DURATION' and 'SUMMARY'. A recording entry shows a duration of '1 day' from '2023-03-13' to '2023-03-14', with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, showing the option 'Order Holter report'. To the right, a device status card for 'Cortrium C3+' displays 'Space' at '4 / 5 GB', 'Charge' at '100%', 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom right of the device card.

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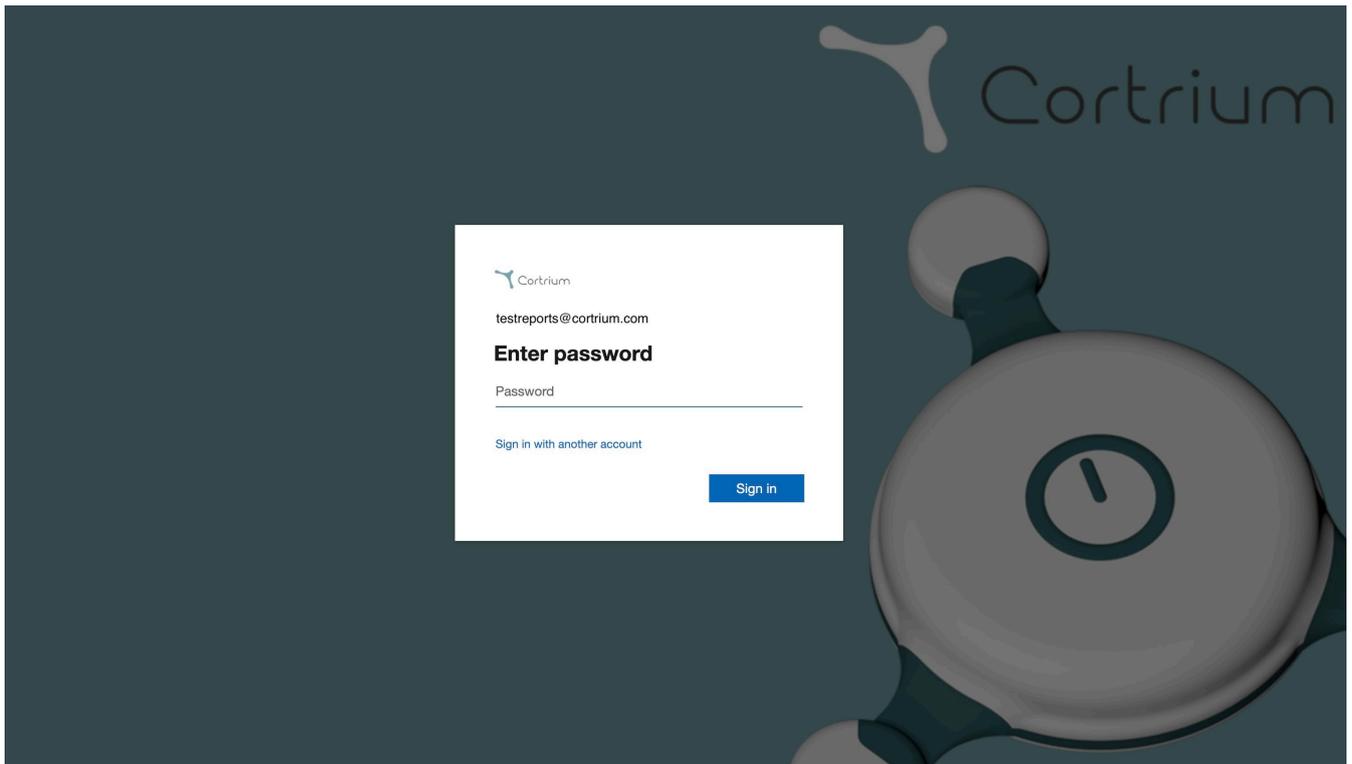
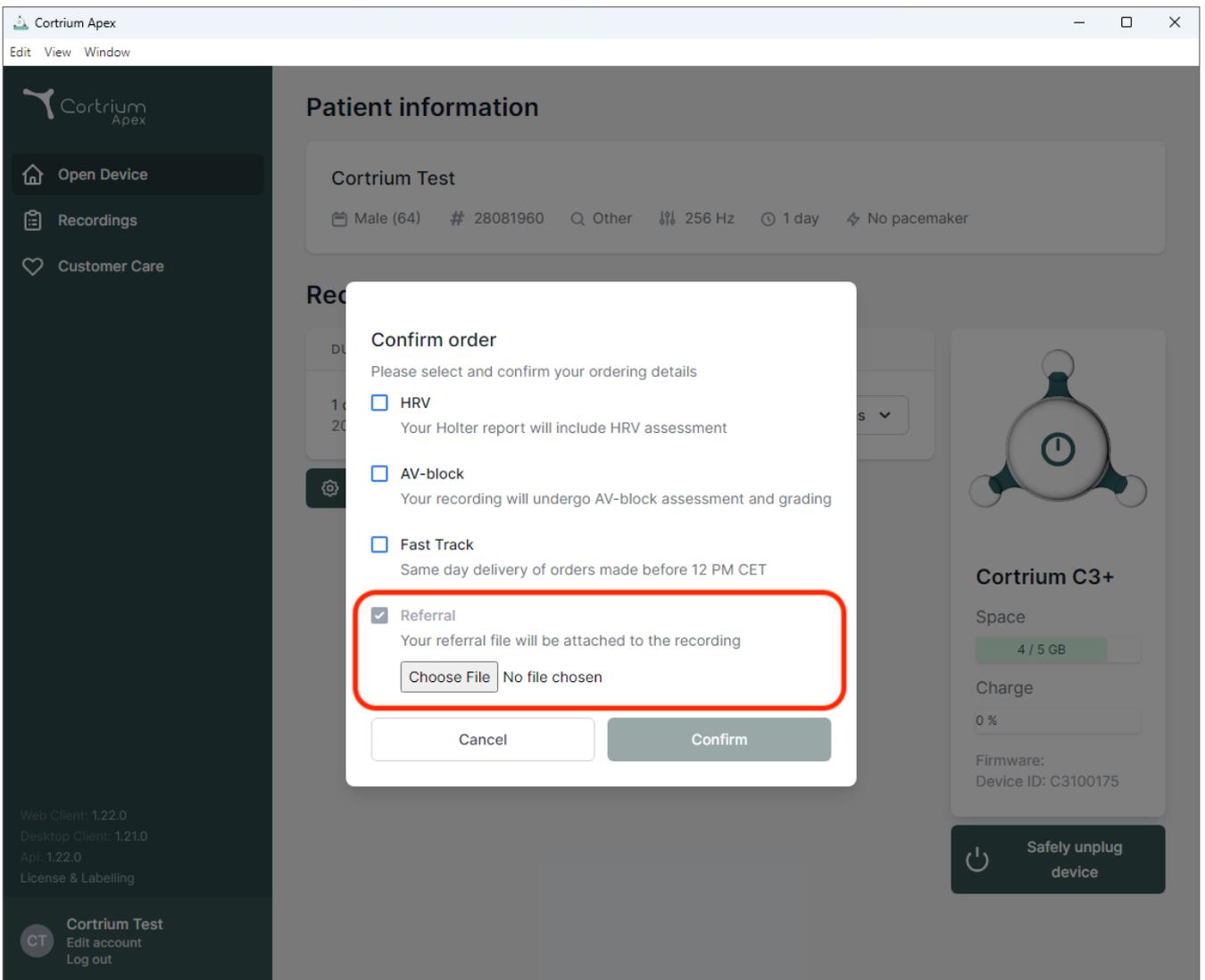
The screenshot displays the Cortrium Apex web application interface. A modal dialog box titled "Confirm order" is centered on the screen, prompting the user to select and confirm their ordering details. The dialog lists four options, each with a checkbox:

- HRV: Your Holter report will include HRV assessment
- AV-block: Your recording will undergo AV-block assessment and grading
- Fast Track: Same day delivery of orders made before 12 PM CET
- Referral: Your referral file will be attached to the recording

The "Referral" option is highlighted with a red border. Below the "Referral" option, there is a "Choose File" button and the text "No file chosen". At the bottom of the dialog, there are "Cancel" and "Confirm" buttons.

The background interface shows the "Patient information" section for "Cortrium Test", including details like "Male (64)", "# 28081960", "Other", "256 Hz", "1 day", and "No pacemaker". On the right, there is a device card for "Cortrium C3+" showing "Space 4 / 5 GB", "Charge 0%", "Firmware: Device ID: C3100175", and a "Safely unplug device" button.

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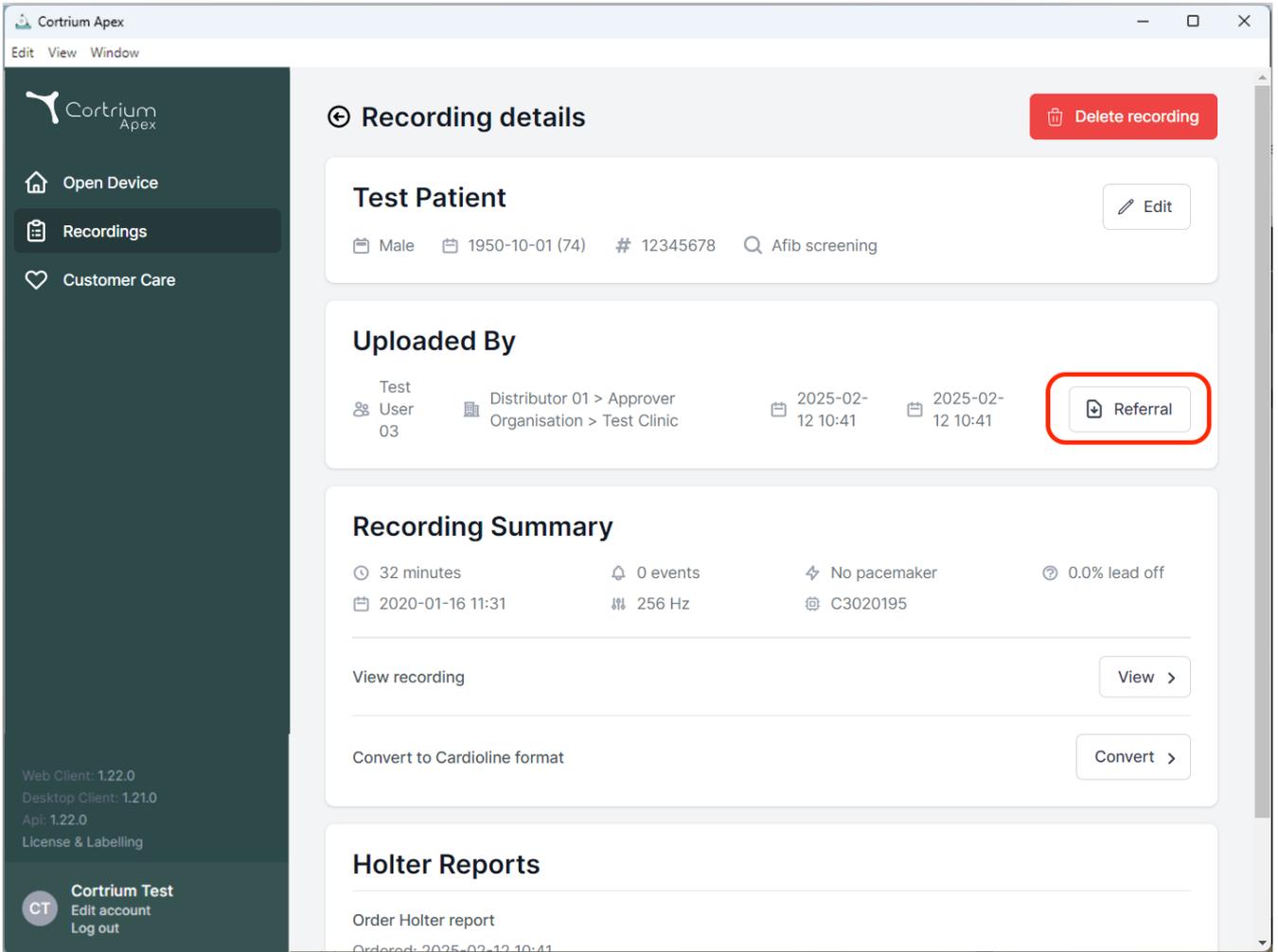
The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is titled 'Recording details' and features a red 'Delete recording' button in the top right corner. Below the title, there are three main sections:

- Test Patient:** Displays patient information: Male, born 1950-10-01 (74), ID # 12345678, and screening type Afib screening. An 'Edit' button is available.
- Uploaded By:** Shows the user 'User 03' from 'Distributor 01 > Approver Organisation > Test Clinic'. The recording was uploaded on 2025-02-12 at 10:41. A 'Referral' button is highlighted with a red box.
- Recording Summary:** Provides key metrics: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date (2020-01-16 11:31) and frequency (256 Hz). Below this, there are buttons for 'View recording' and 'Convert to Cardioline format'.

At the bottom, the 'Holter Reports' section is partially visible, showing an 'Order Holter report' button and the text 'Ordered: 2025-02-12 10:41'.

Footer information in the sidebar includes: Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling, and user 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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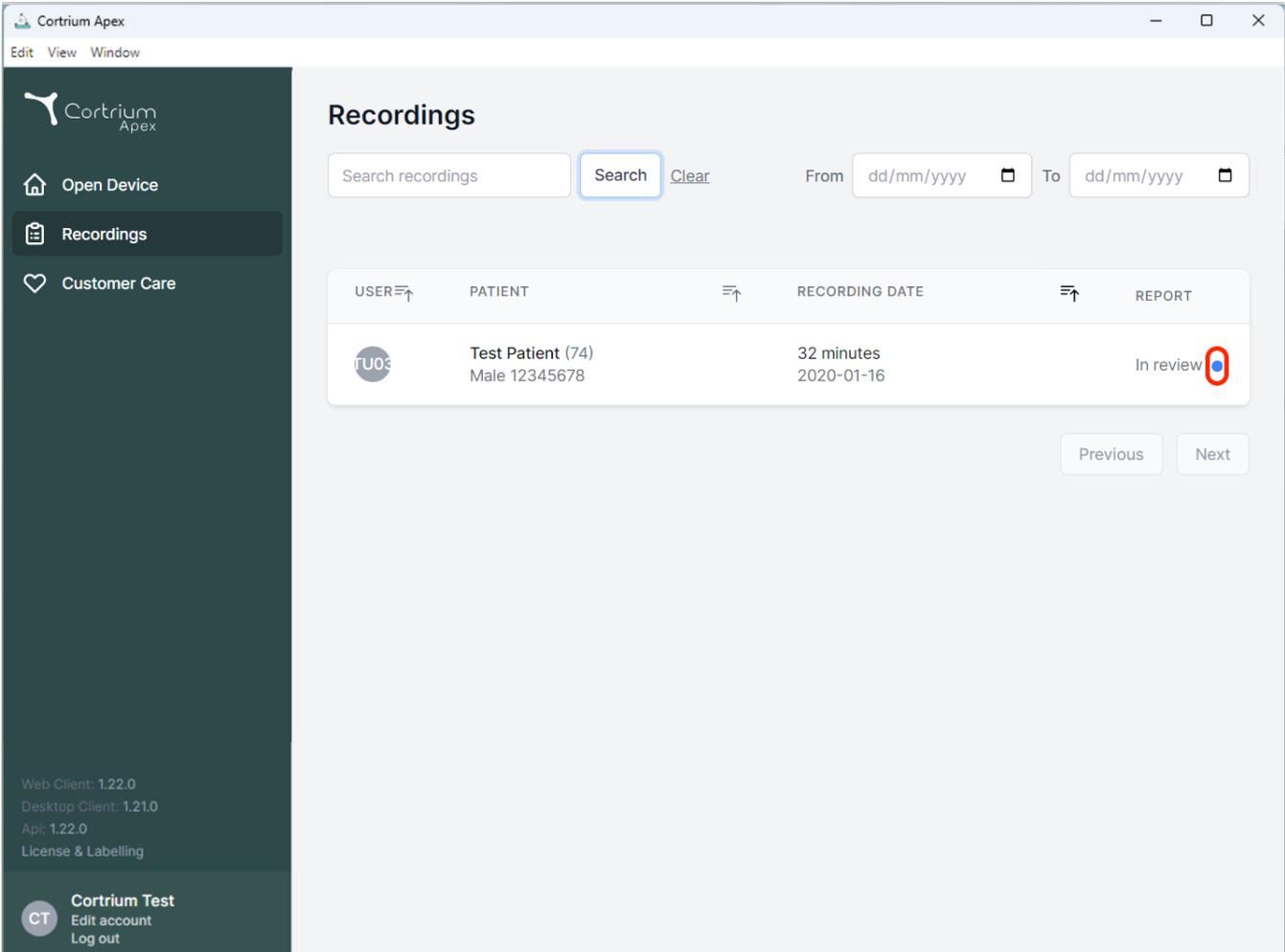### Recordings

Search recordings    From   To

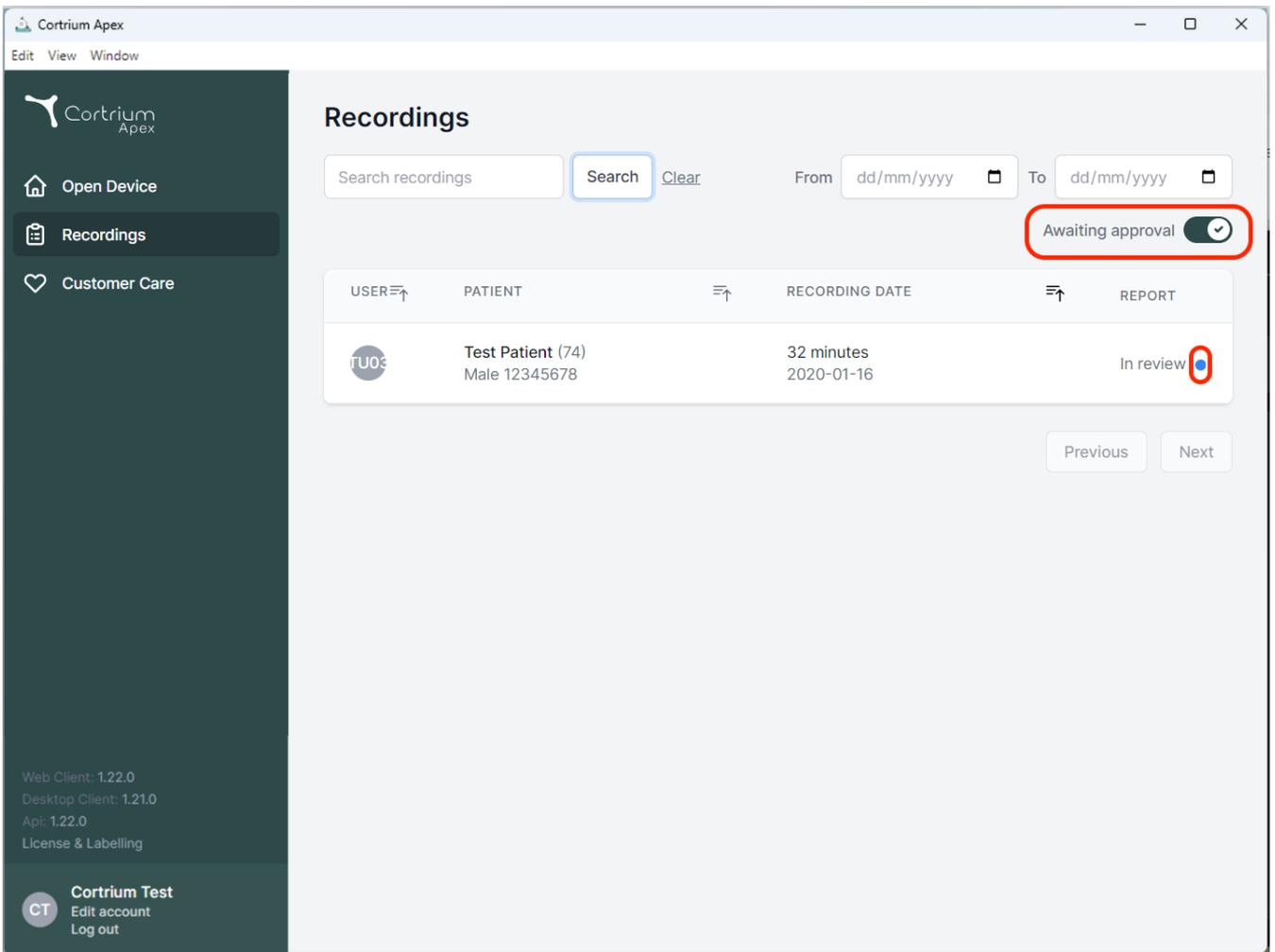
USER ↕	PATIENT	RECORDING DATE	REPORT
	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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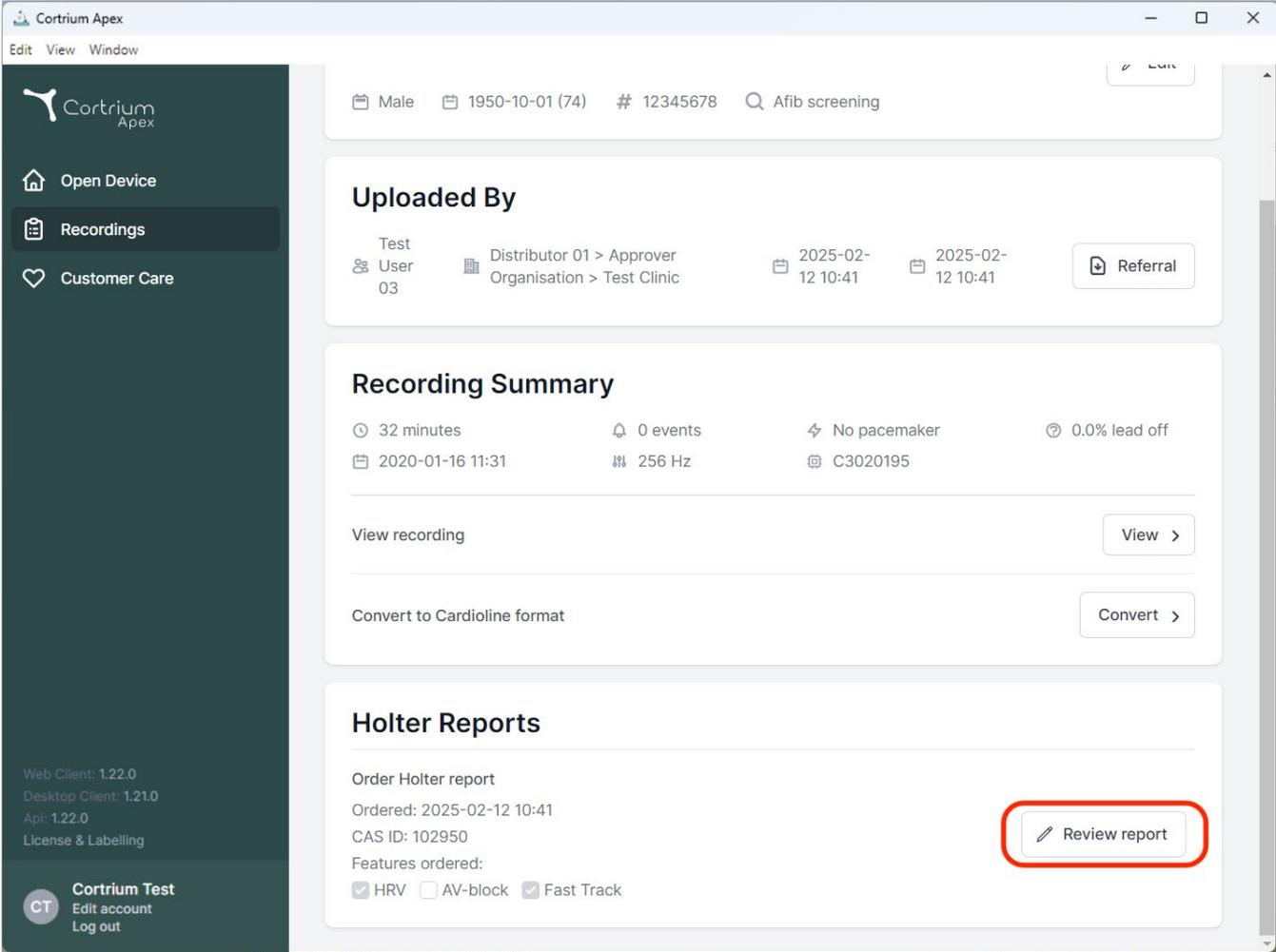
Search recordings  Search  From  To

Awaiting approval

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the 'Uploaded By' section, showing 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41 and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and recording date 2020-01-16 11:31. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report' with 'Ordered: 2025-02-12 10:41' and 'CAS ID: 102950'. A 'Review report' button is highlighted with a red box. The bottom left corner shows version information and user 'Cortrium Test' with 'Edit account' and 'Log out' options.

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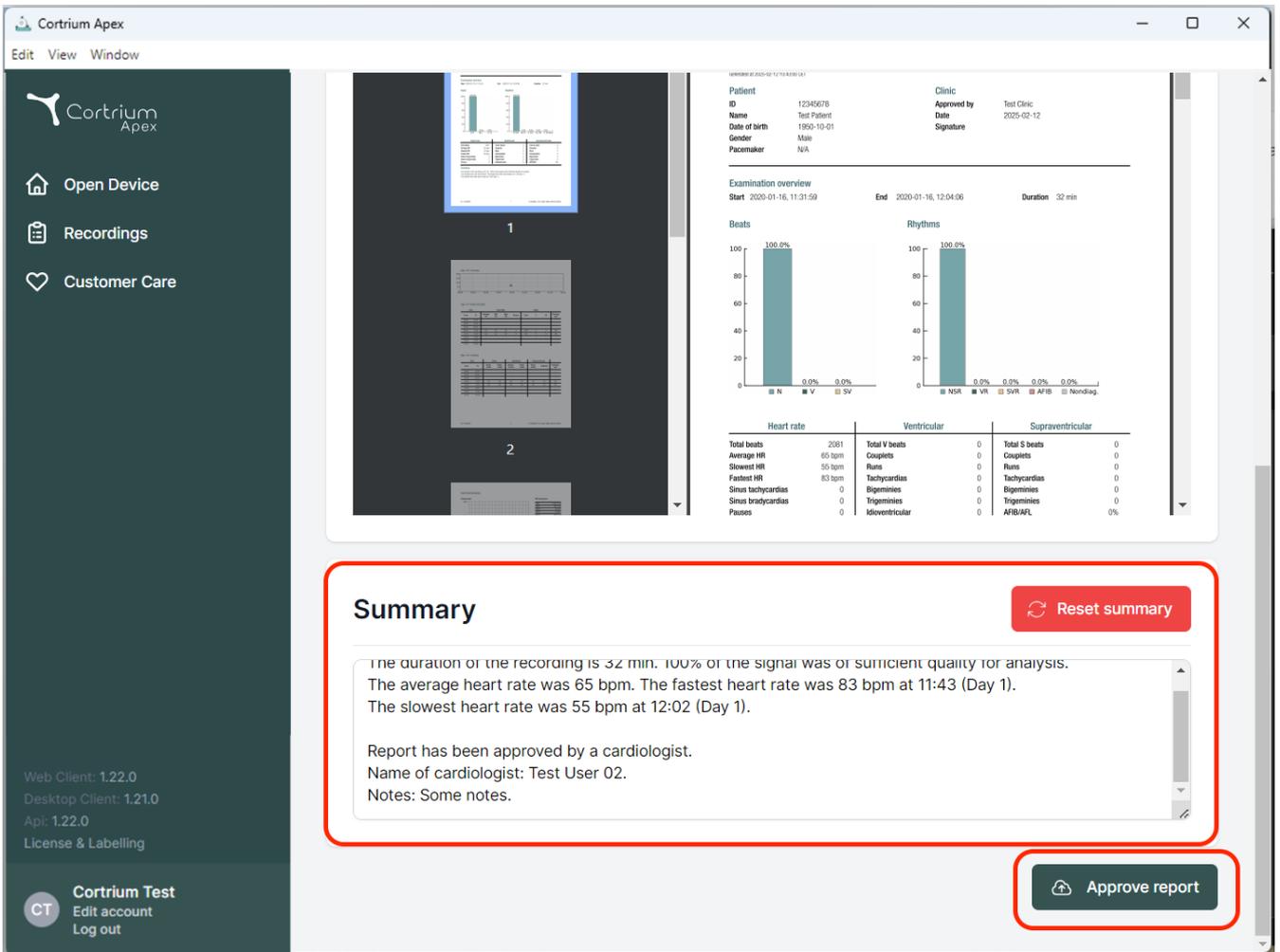The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Patient Information:** ID 12345678, Name Test Patient, Date of birth 1950-10-01, Gender Male, Pacemaker N/A. Clinic: Test Clinic, Approved by Test Clinic, Date 2025-02-12.
- Examination overview:** Start 2020-01-16, 11:31:59, End 2020-01-16, 12:04:06, Duration 32 min.
- Beats and Rhythms:** Two bar charts showing 100.0% for NSR (Normal Sinus Rhythm) and 0.0% for other categories (V, SV, NSR, VR, SVR, AFIB, Nondiag).
- Summary Table:**

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigemines	0	Bigemines	0
Sinus bradycardias	0	Trigemines	0	Trigemines	0
Pauses	0	Idioventricular	0	AFIB/AFL	0%
- Summary Section (highlighted with a red box):**

**Summary** Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.  
Name of cardiologist: Test User 02.  
Notes: Some notes.

Approve report

At the bottom left, version information is shown: Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling. The bottom right corner shows 'Cortrium Test', 'Edit account', and 'Log out'.

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark  
E-mail : [info@cortrium.com](mailto:info@cortrium.com) • Site Web : [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033