Cortrium Apex 1.22

Istruzioni d'uso

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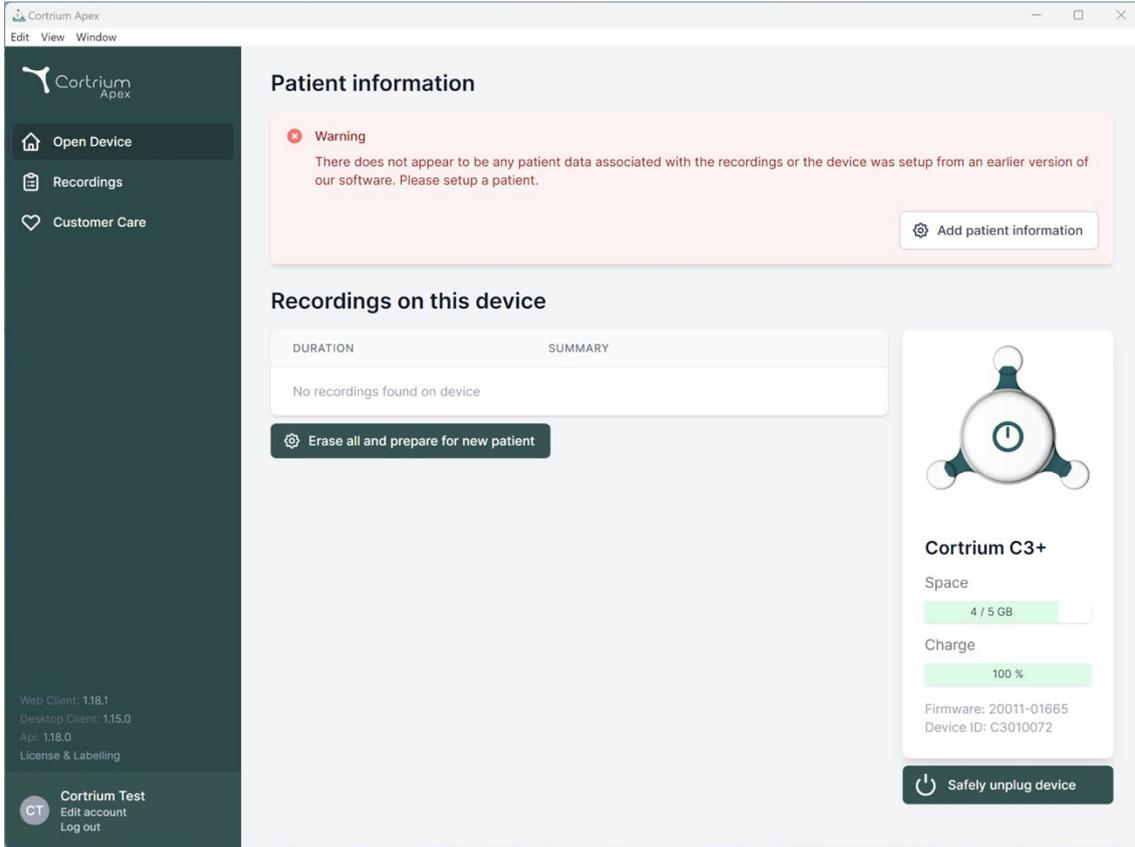
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, it shows version information (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and user options (Cortrium Test, Edit account, Log out). The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' Below this is a table for 'Recordings on this device' with columns for 'DURATION' and 'SUMMARY', showing 'No recordings found on device'. A button 'Erase all and prepare for new patient' is located below the table. On the right, a device status card for 'Cortrium C3+' shows 'Space: 4 / 5 GB' and 'Charge: 100%'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072', with a 'Safely unplug device' button at the bottom.

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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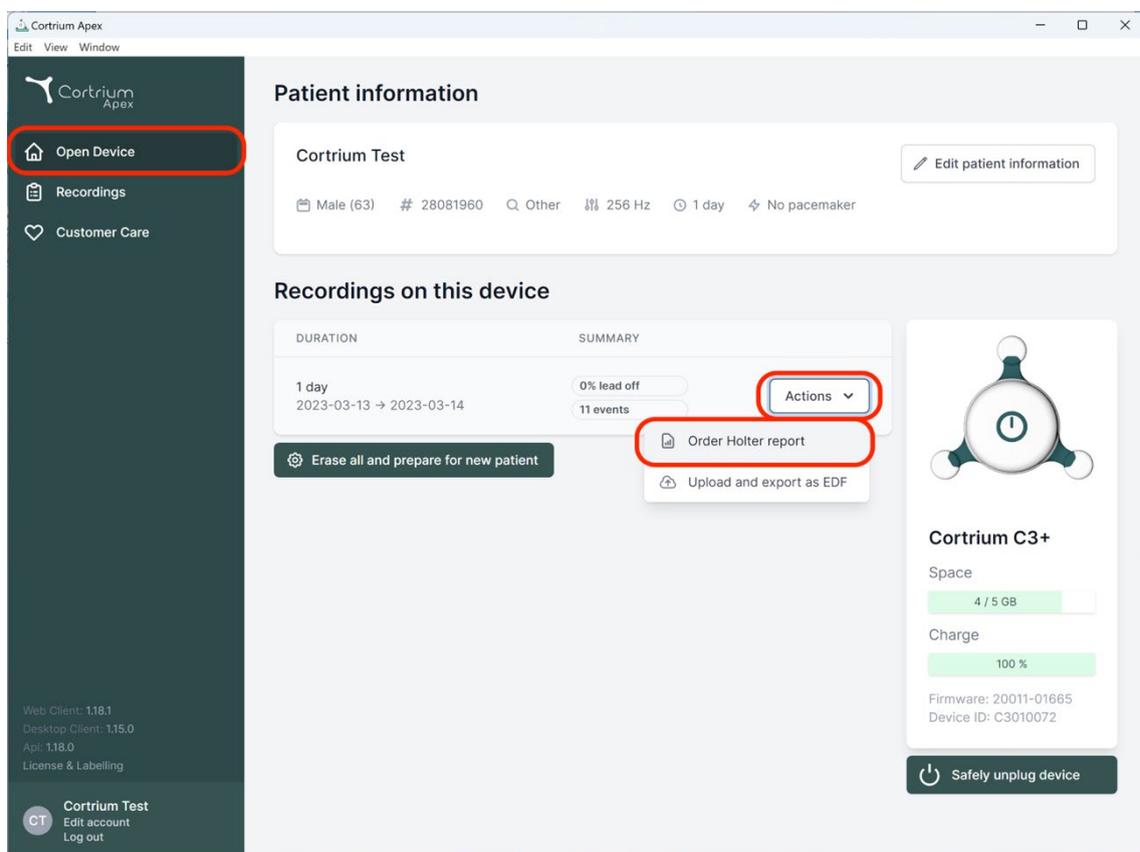
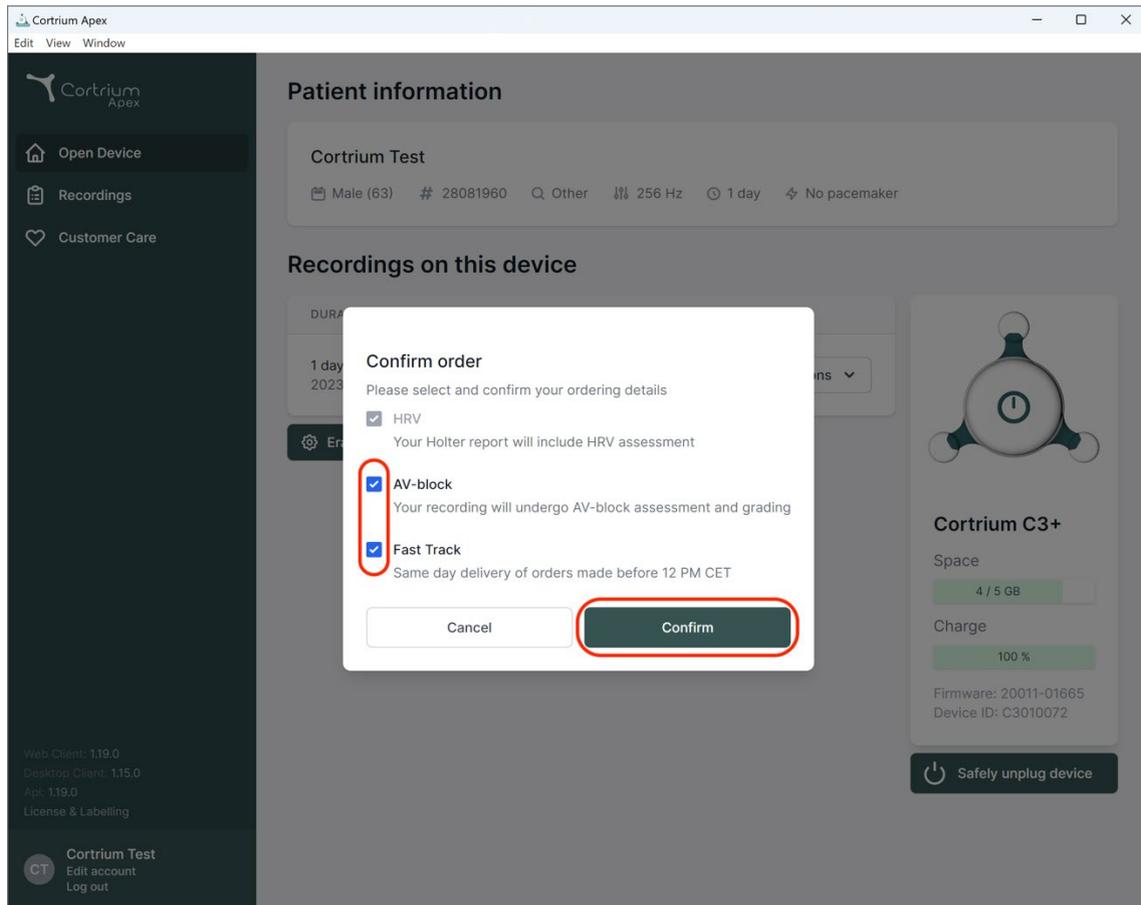
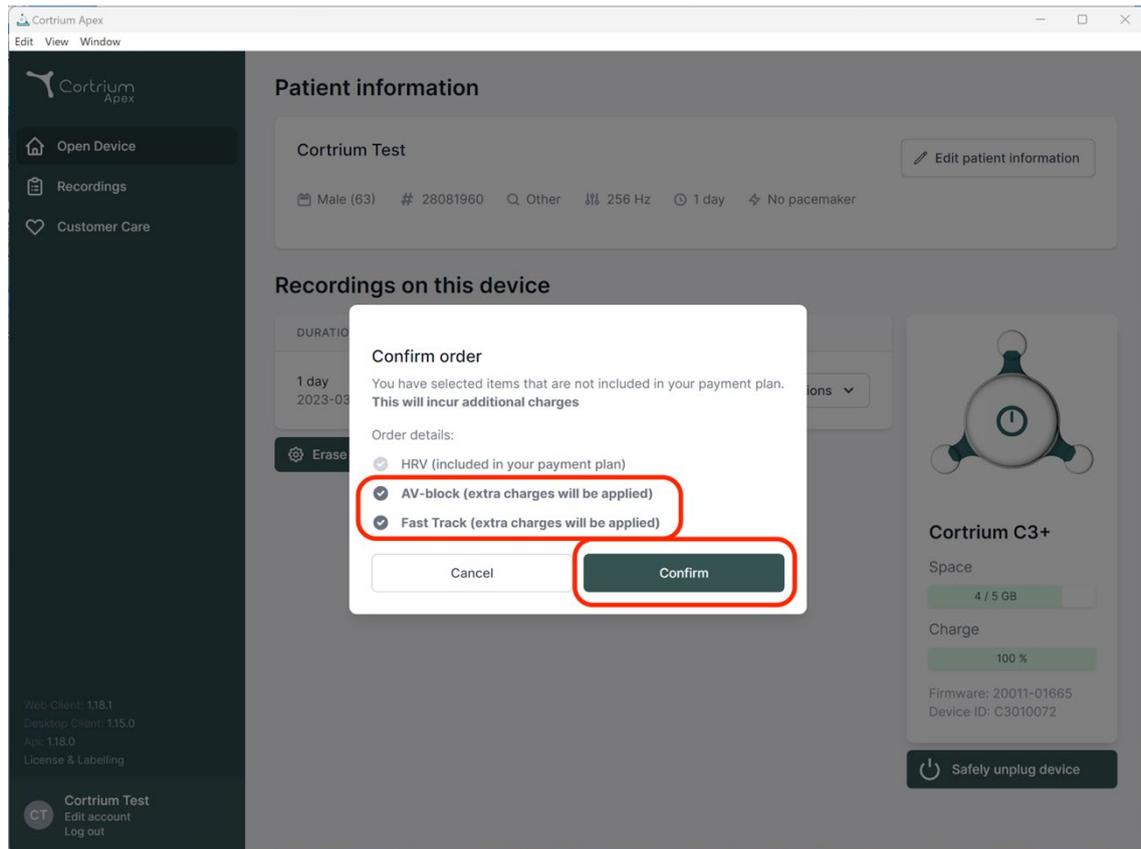
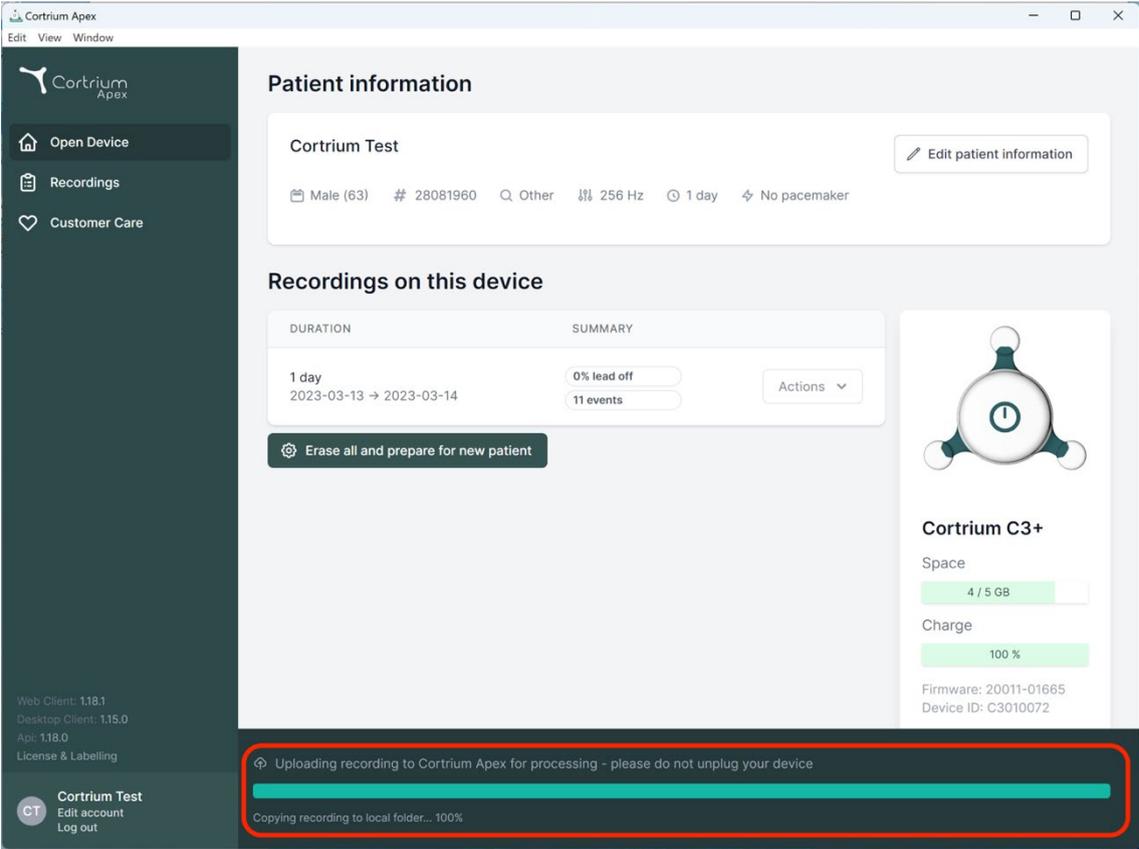
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a recording summary with '0% lead off' and '11 events'. A 'Cortrium C3+' device status panel on the right indicates 4/5 GB of space used and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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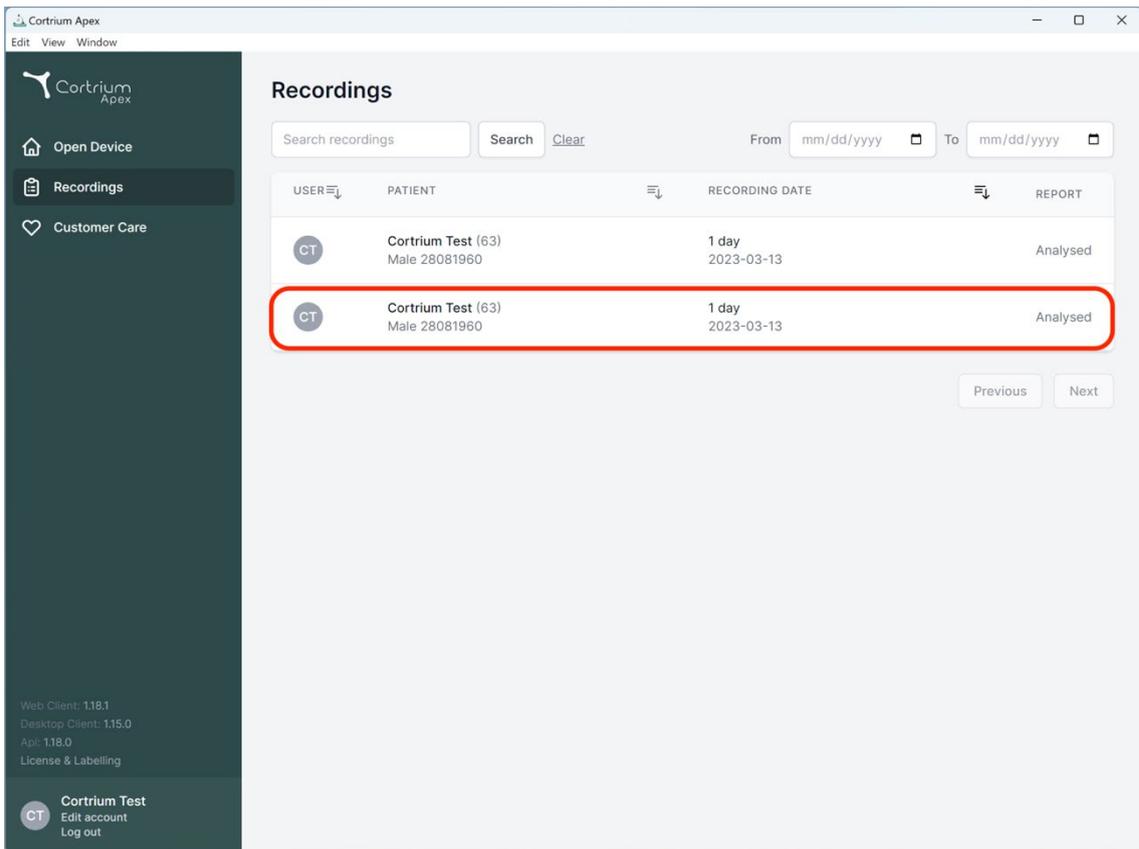
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area features a search bar, date filters, and a table of recordings. The table has columns for 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. Two rows are visible, both for 'Cortrium Test (63)' with patient ID 'Male 28081960', recording date '2023-03-13', and report status 'Analysed'. The second row is highlighted with a red border. At the bottom of the table, there are 'Previous' and 'Next' navigation buttons. The sidebar also shows version information and a user profile for 'Cortrium Test'.

| USER | PATIENT | RECORDING DATE | REPORT |
|------|-------------------------------------|---------------------|----------|
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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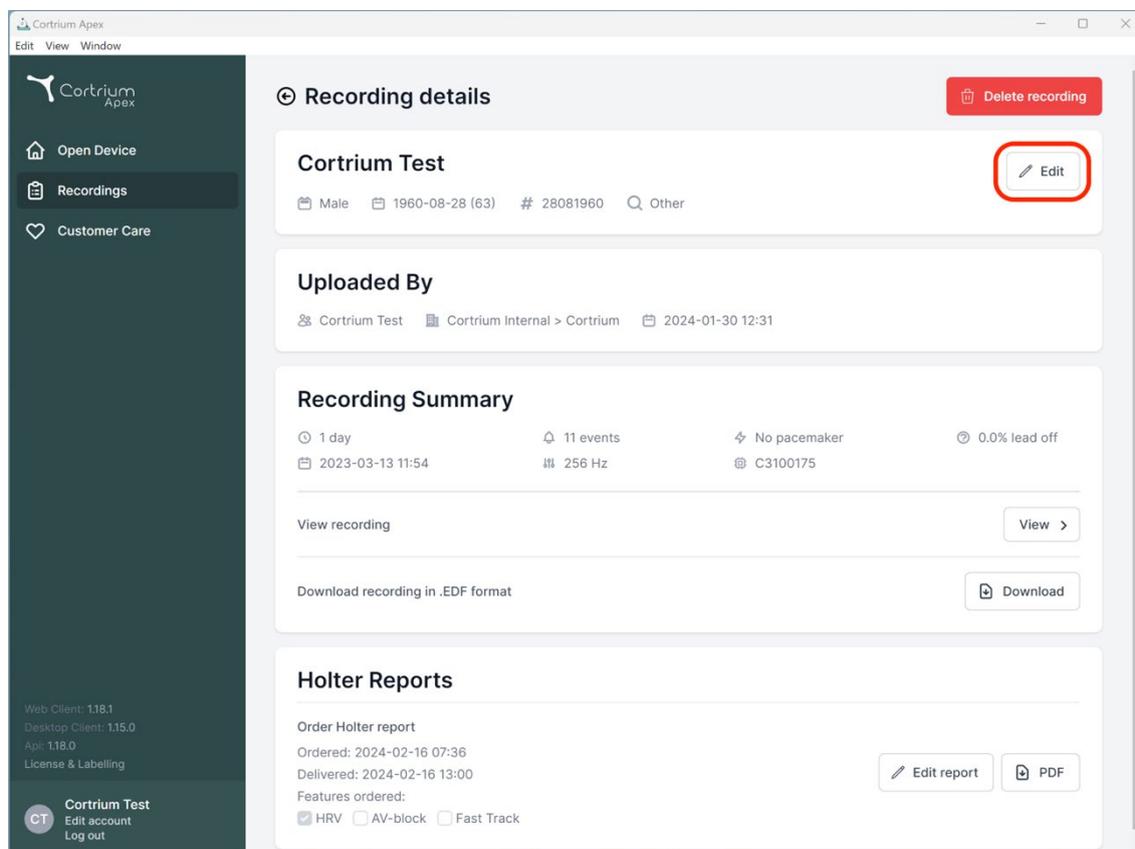
Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

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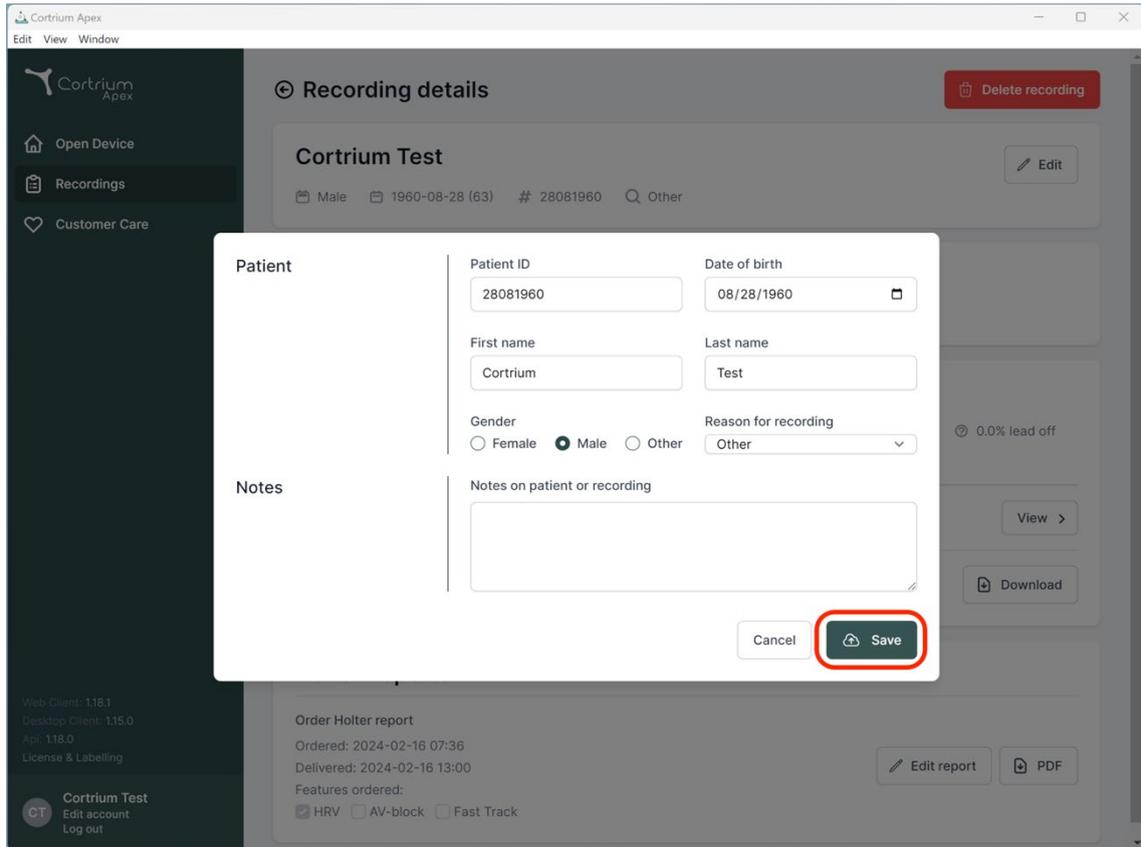
Holter Reports

Order Holter report
Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00 Edit report PDF

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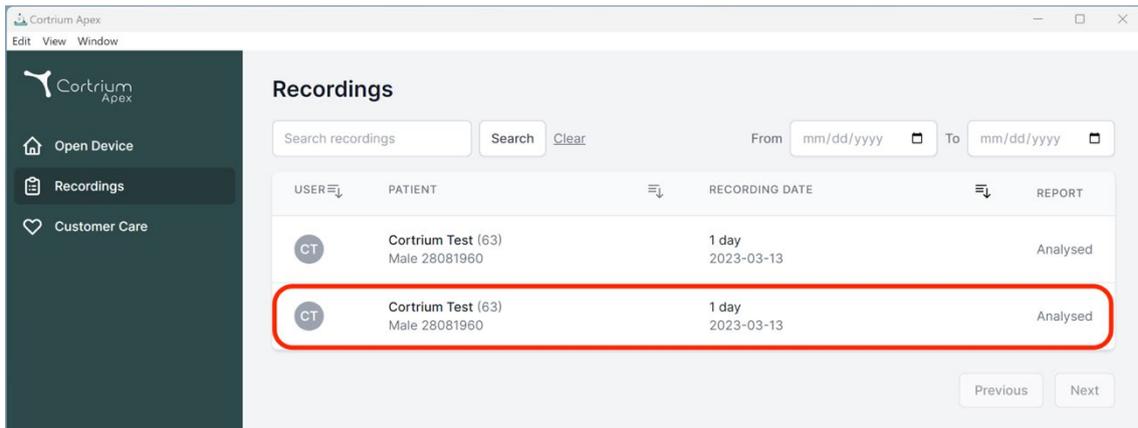
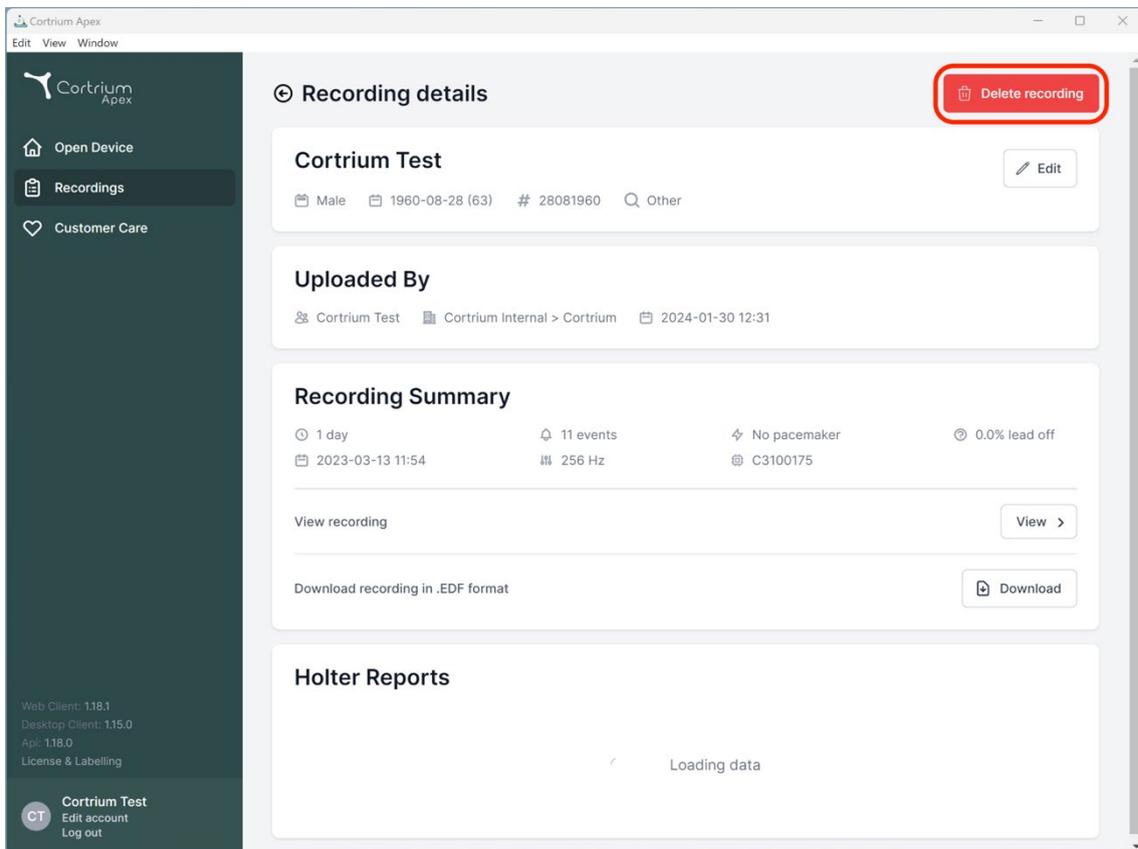
Recording details Delete recording

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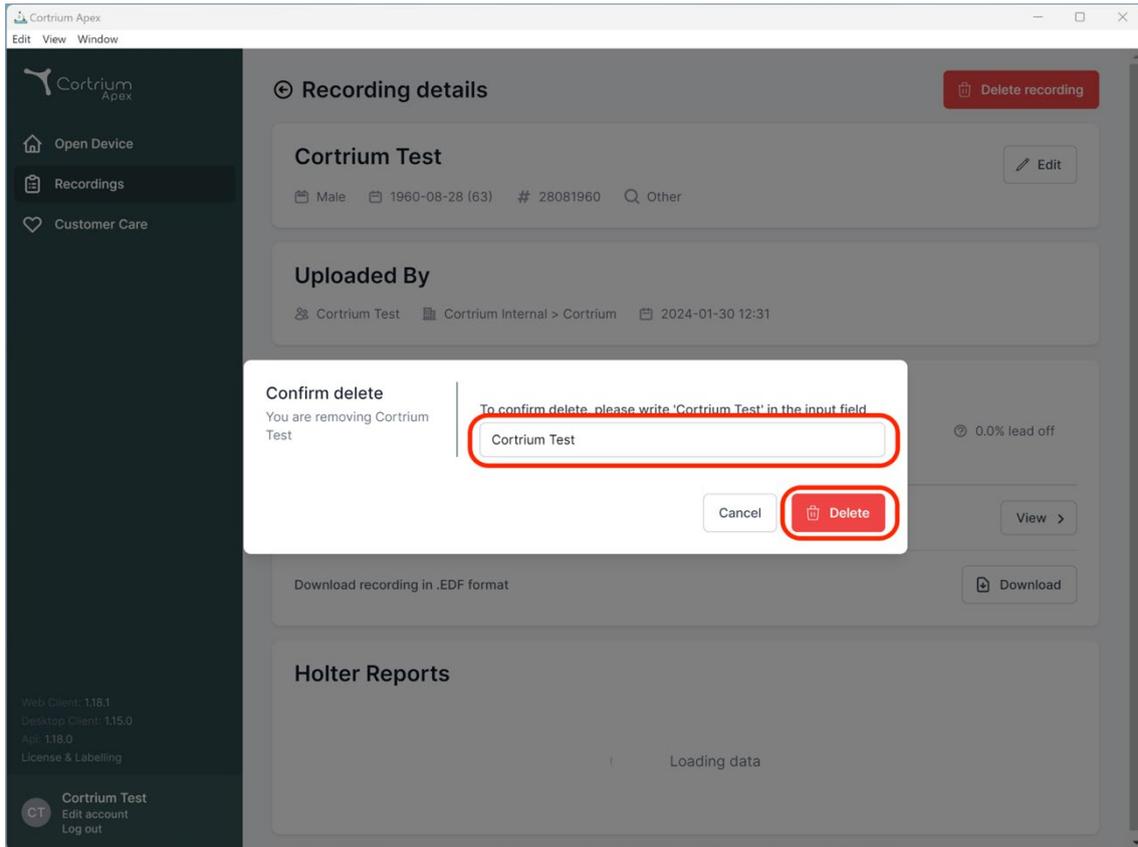
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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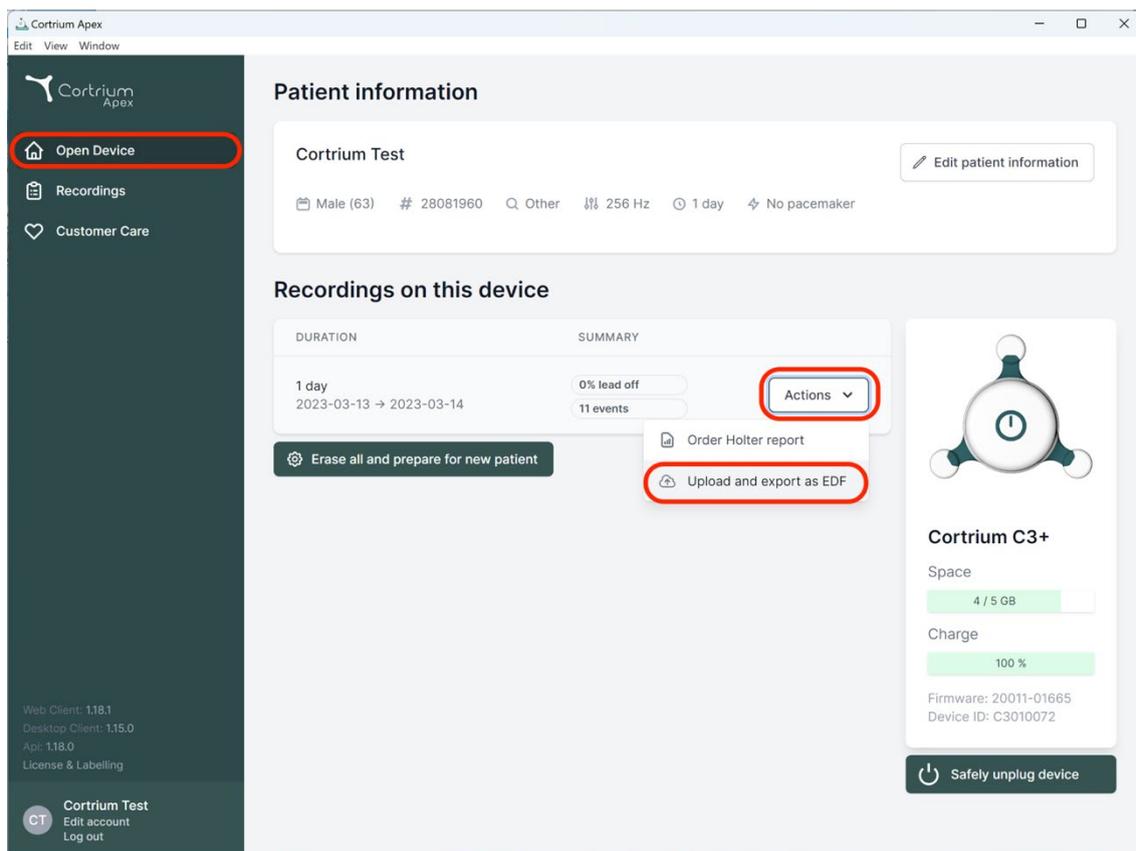
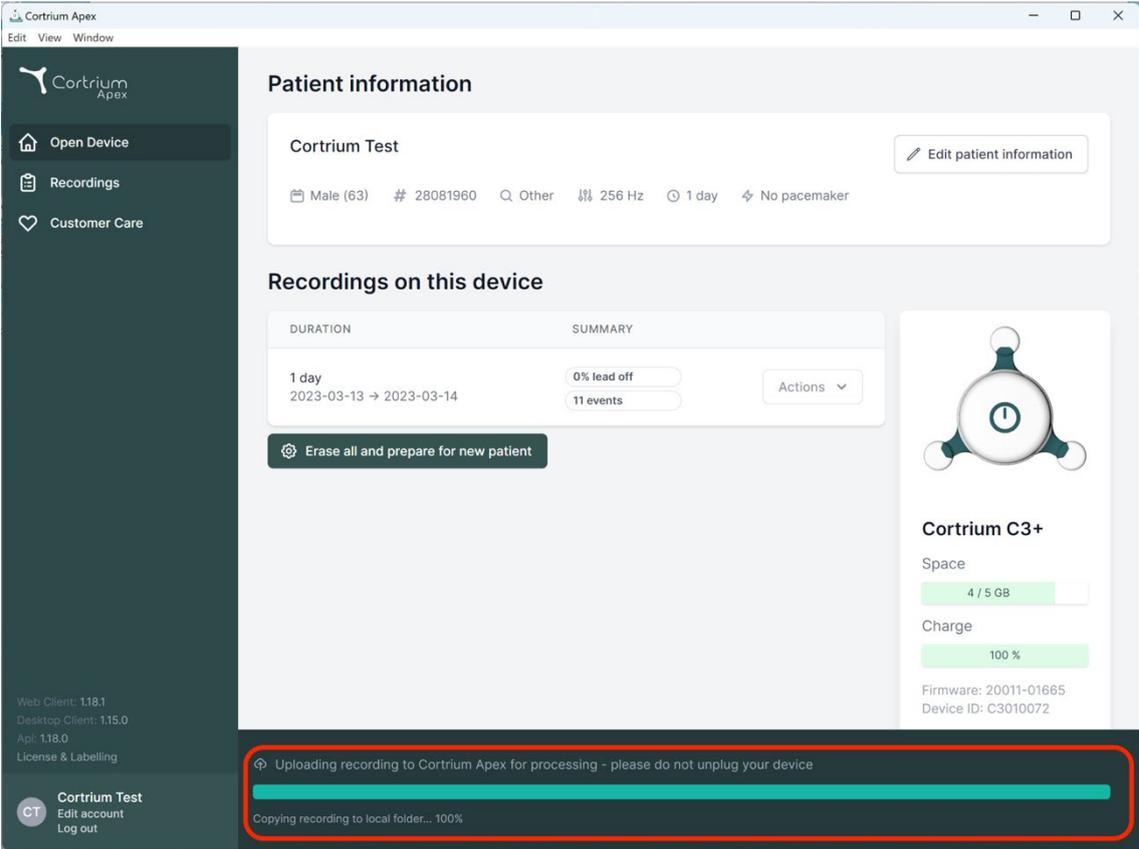
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY | |
|----------------------------------|--------------------------|-------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events | Actions |

[Erase all and prepare for new patient](#)

Uploading recording to Cortrium Apex for processing - please do not unplug your device

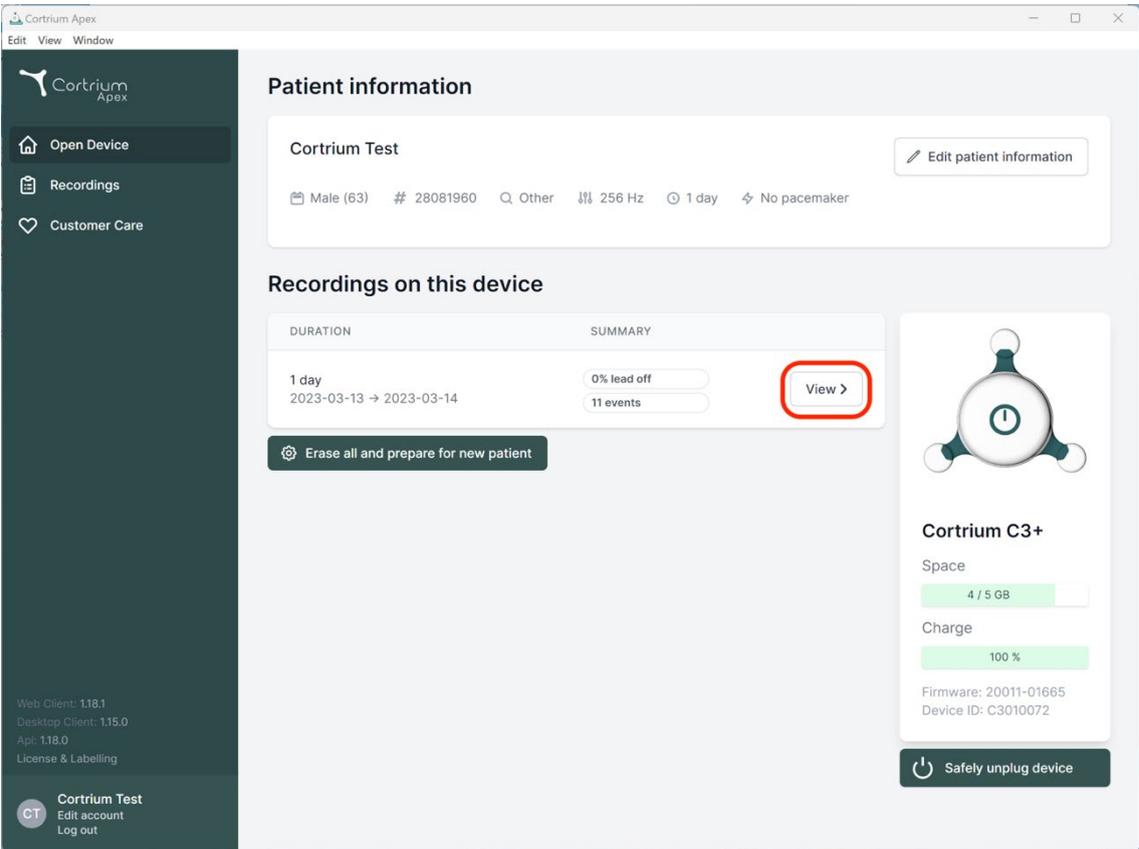
Copying recording to local folder... 100%

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY | |
|----------------------------------|--------------------------|---------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events | View > |

[Erase all and prepare for new patient](#)

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
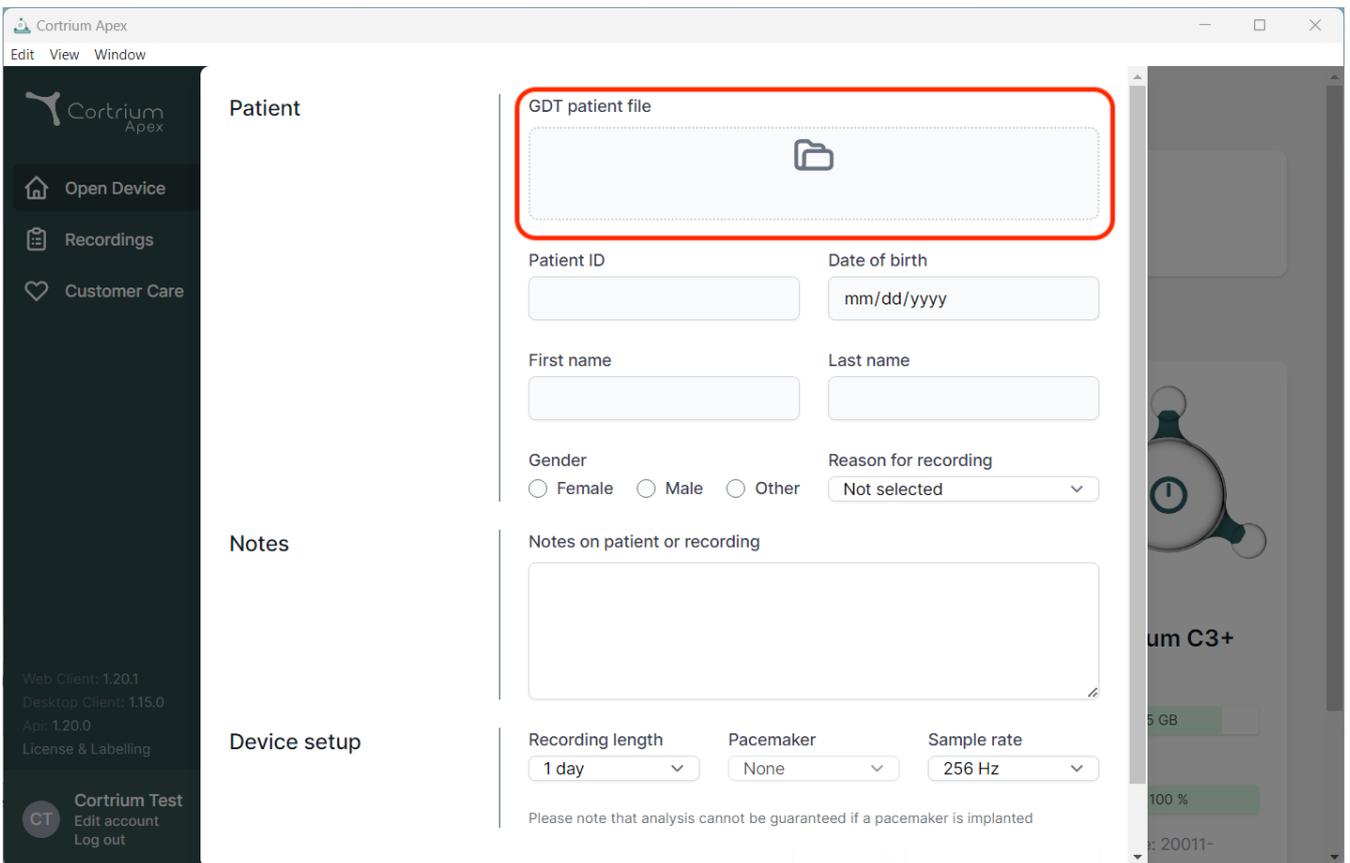
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" input field, which is currently empty and contains a folder icon. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently set to "Not selected"). Below these fields is a "Notes" section with a text area labeled "Notes on patient or recording". At the bottom of the form is the "Device setup" section, which includes "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below the device setup section states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar shows version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". The bottom of the sidebar also shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

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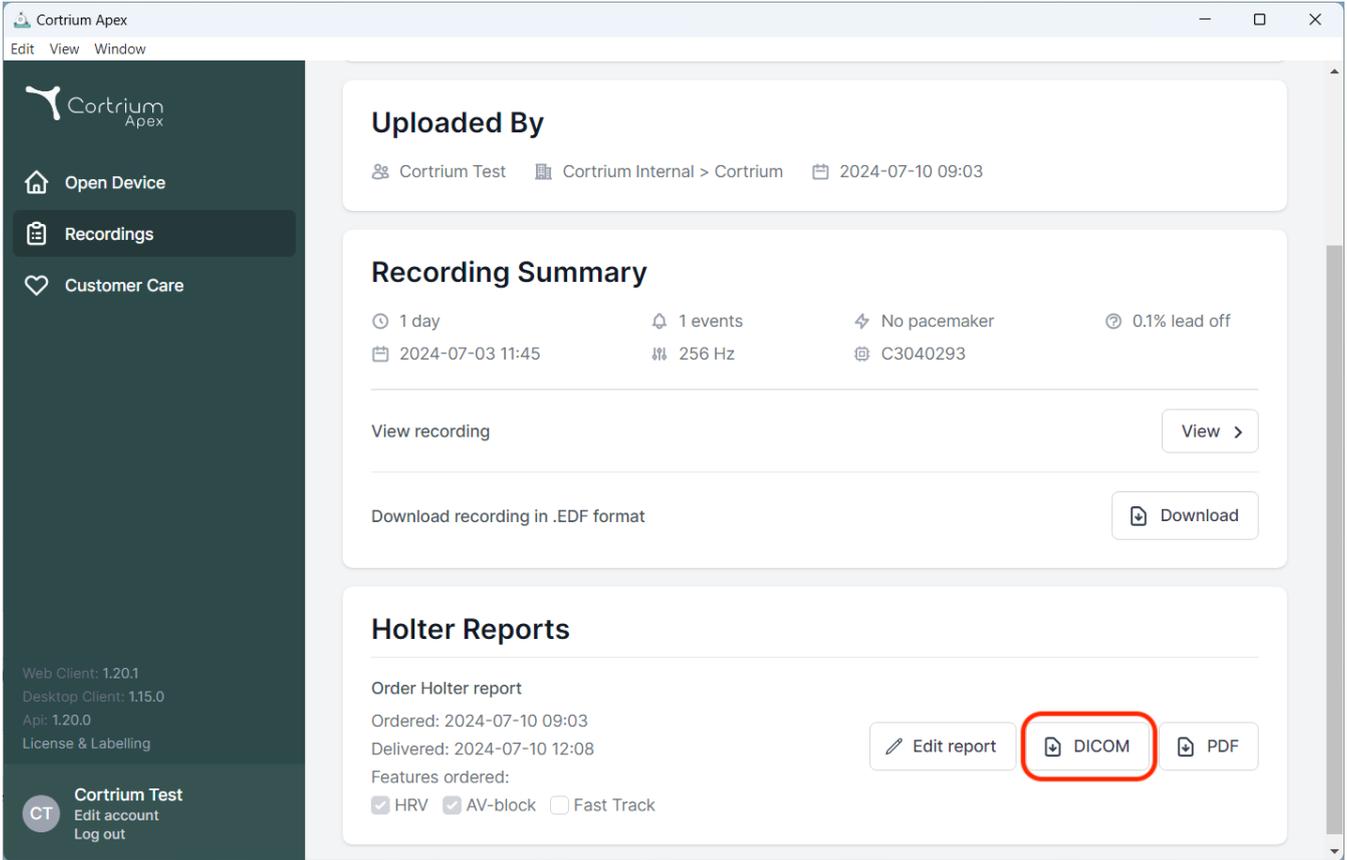
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered' as 'HRV', 'AV-block', and 'Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. System version information is also visible: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', and 'Api: 1.20.0'.

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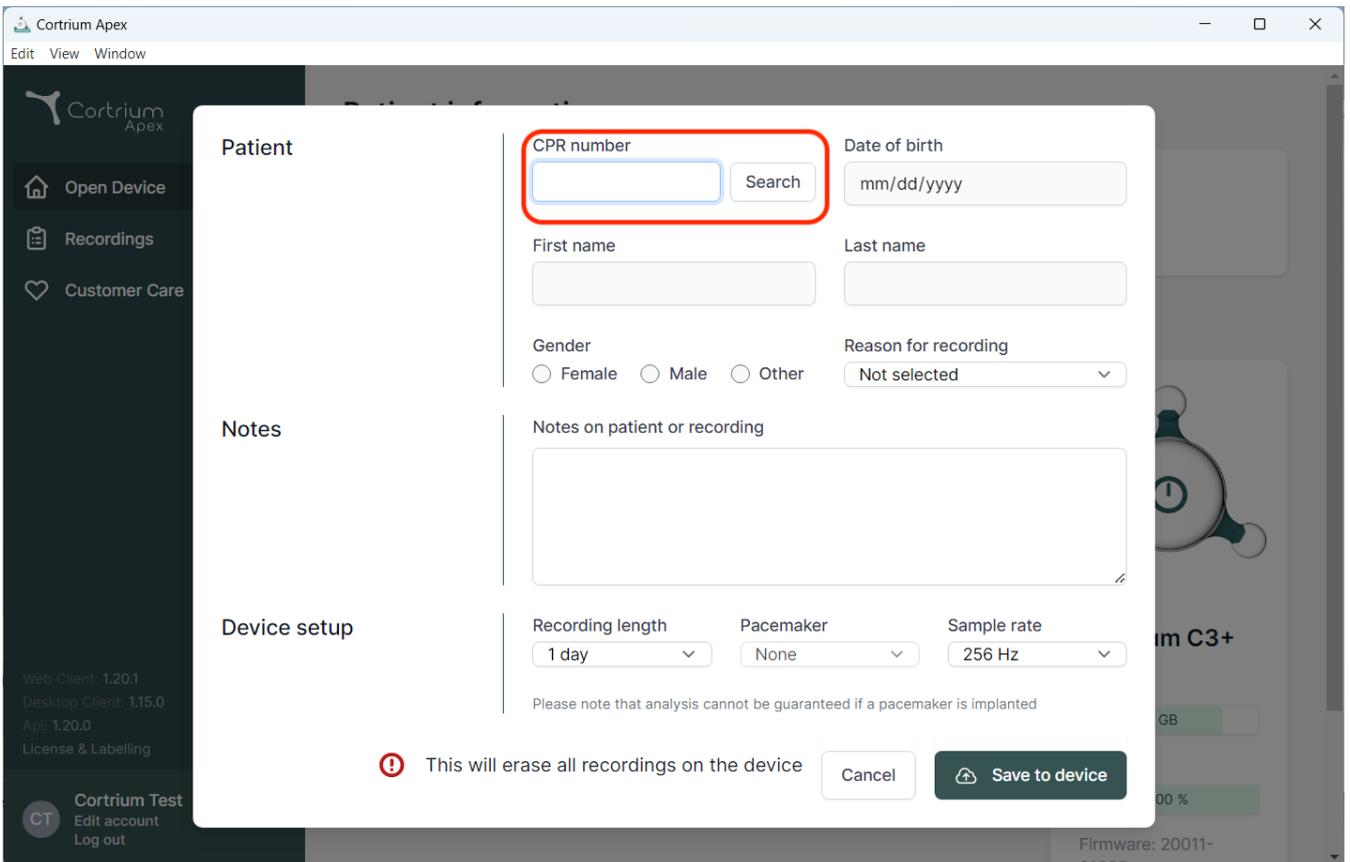
The screenshot displays the 'Cortrium Apex' web application interface. A modal window titled 'Patient' is open, containing the following fields and controls:

- CPR number:** A text input field with a 'Search' button next to it, highlighted by a red box.
- Date of birth:** A text input field with the placeholder 'mm/dd/yyyy'.
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for 'Female', 'Male', and 'Other'.
- Reason for recording:** A dropdown menu currently set to 'Not selected'.
- Notes:** A large text area labeled 'Notes on patient or recording'.
- Device setup:** Three dropdown menus for 'Recording length' (set to '1 day'), 'Pacemaker' (set to 'None'), and 'Sample rate' (set to '256 Hz').

Below the device setup section, a warning message with a red exclamation mark icon reads: 'This will erase all recordings on the device'. At the bottom right of the modal, there are 'Cancel' and 'Save to device' buttons.

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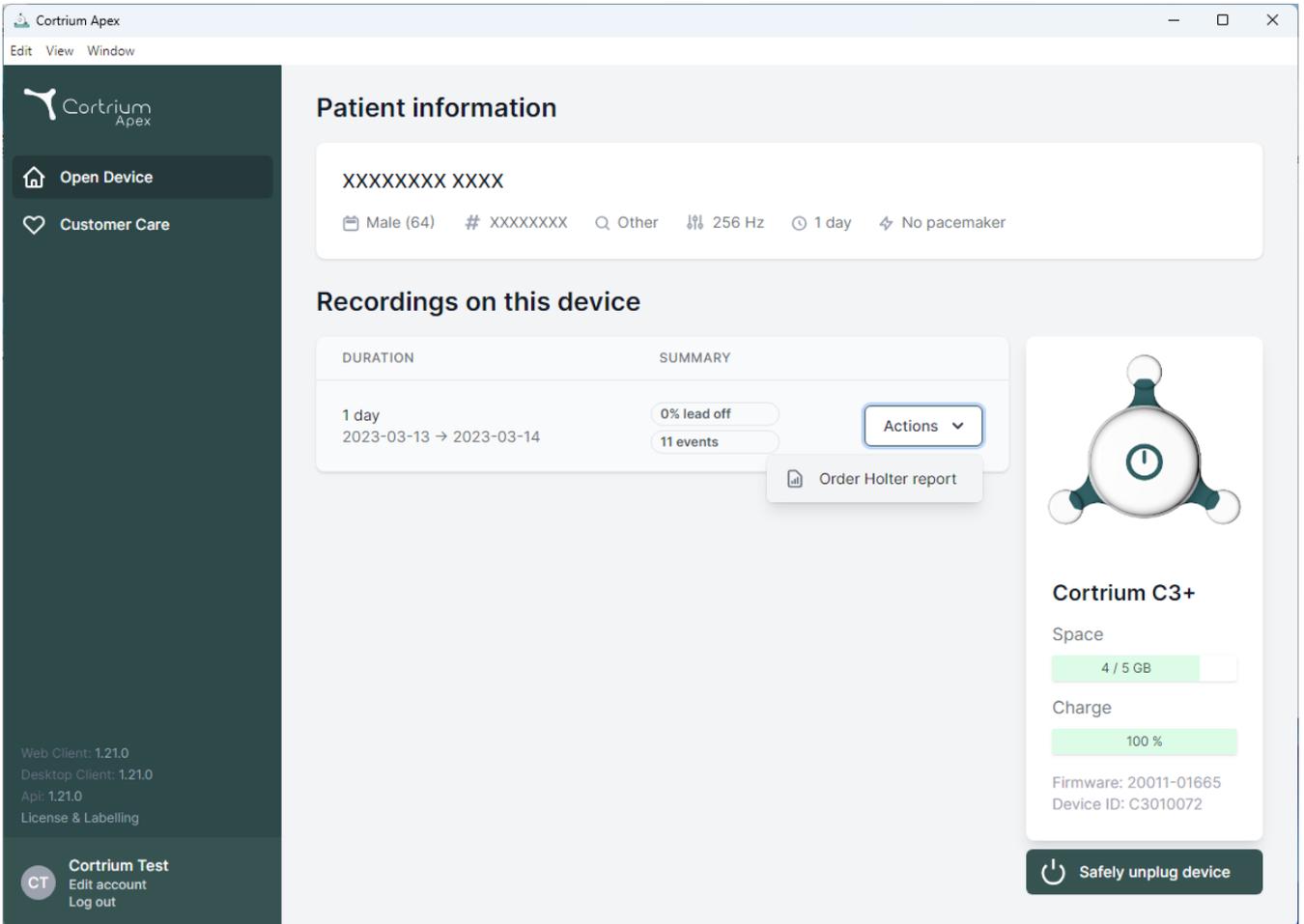The screenshot displays the Cortrium Apex web application interface. The browser window title is "Cortrium Apex". The interface includes a dark sidebar on the left with the Cortrium Apex logo and navigation options: "Open Device" and "Customer Care". At the bottom of the sidebar, it shows version information: "Web Client: 1.21.0", "Desktop Client: 1.21.0", "Api: 1.21.0", and "License & Labelling". A user profile section at the bottom left identifies the user as "Cortrium Test" with options to "Edit account" and "Log out".

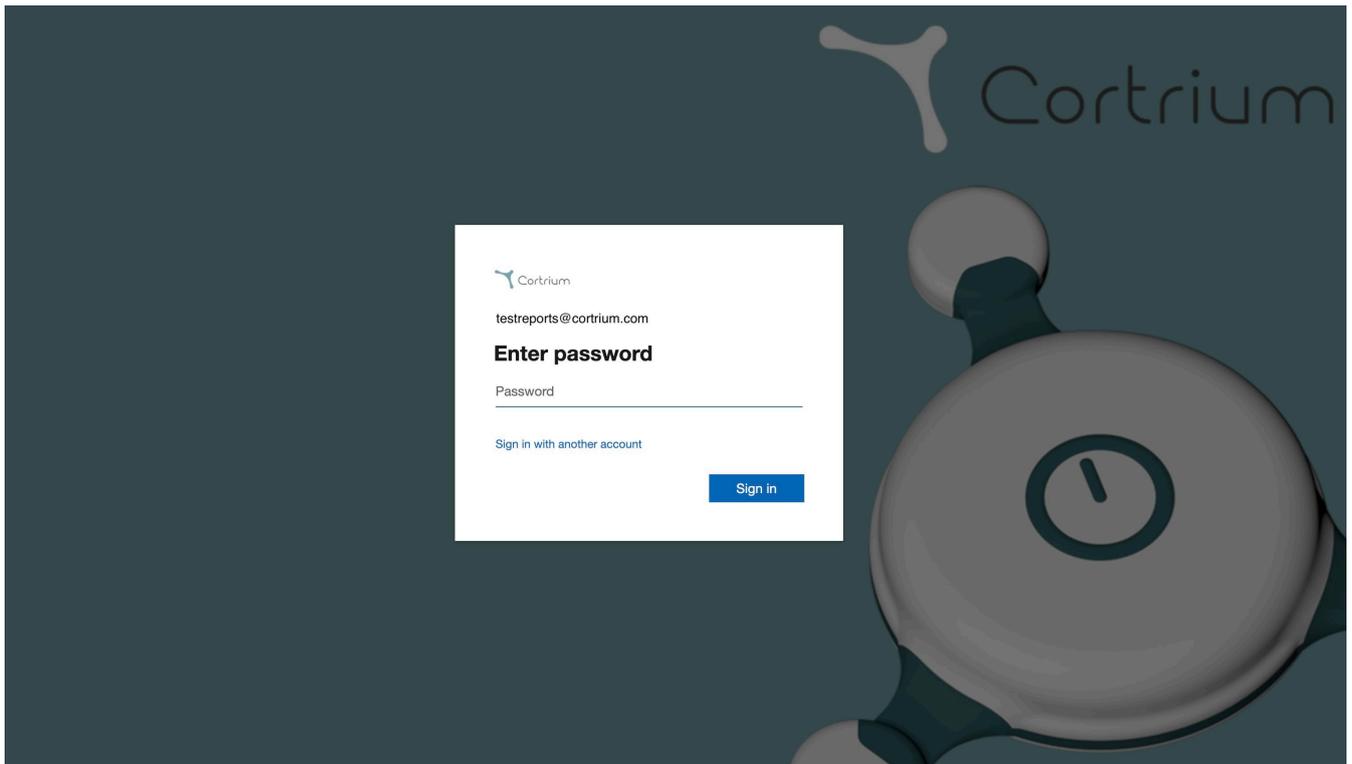
The main content area is titled "Patient information" and shows a patient ID "XXXXXXXX XXXX". Below this, patient details are listed: "Male (64)", "# XXXXXXXX", "Other", "256 Hz", "1 day", and "No pacemaker".

The "Recordings on this device" section features a table with two columns: "DURATION" and "SUMMARY". A recording entry is shown for "1 day" from "2023-03-13" to "2023-03-14". The summary indicates "0% lead off" and "11 events". An "Actions" dropdown menu is open, showing the option "Order Holter report".

On the right side, there is a device status card for "Cortrium C3+". It displays a battery icon, "Space" usage at "4 / 5 GB", and "Charge" at "100%". It also lists "Firmware: 20011-01665" and "Device ID: C3010072". A "Safely unplug device" button is located at the bottom of this card.

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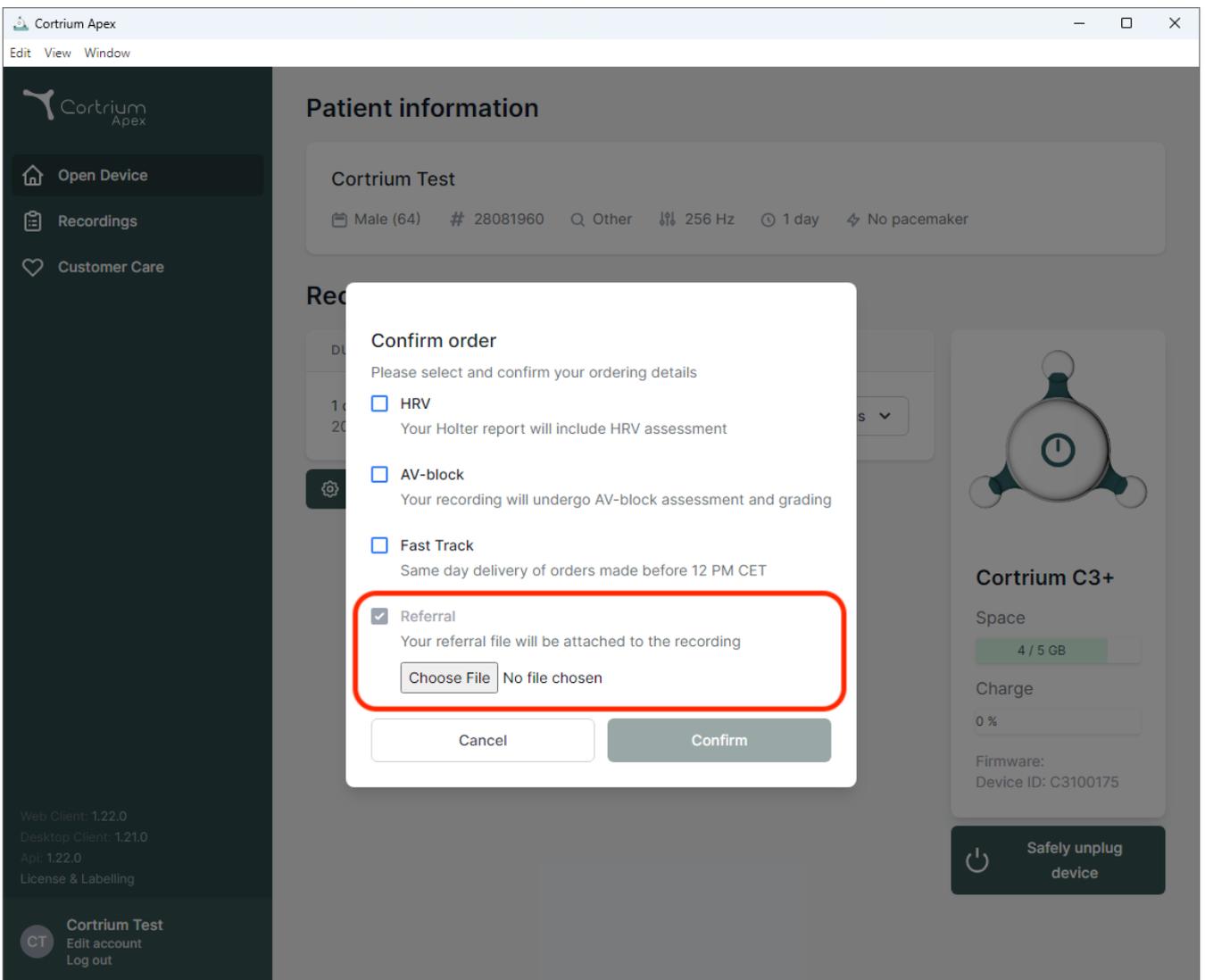
Male (64) # 28081960 Other 256 Hz 1 day No pacemaker

Confirm order

Please select and confirm your ordering details

- HRV
Your Holter report will include HRV assessment
- AV-block
Your recording will undergo AV-block assessment and grading
- Fast Track
Same day delivery of orders made before 12 PM CET
- Referral
Your referral file will be attached to the recording
 No file chosen

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
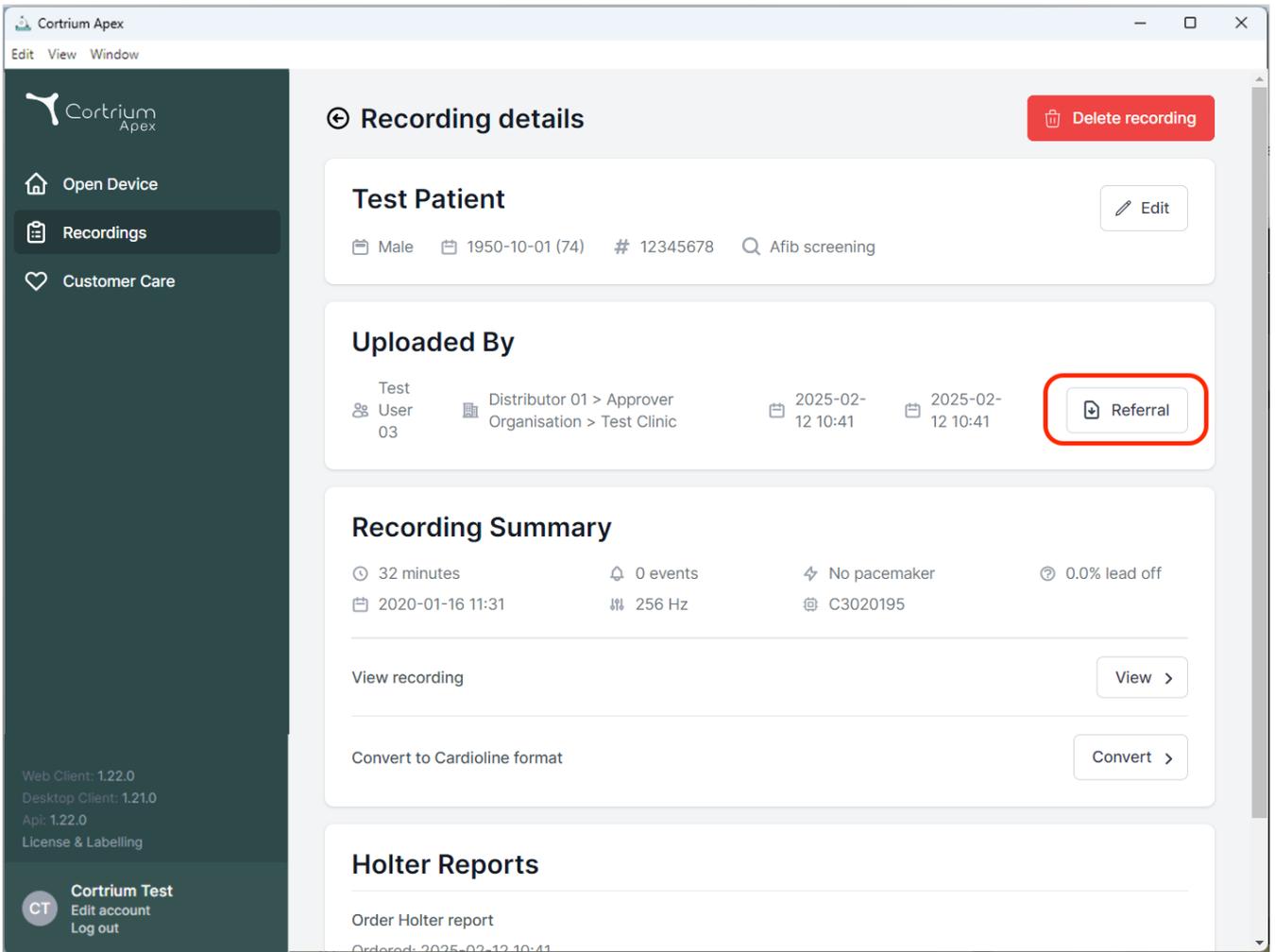
The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right corner. Below this, the 'Test Patient' section shows details such as 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening', with an 'Edit' button. The 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic', along with upload dates of '2025-02-12 10:41'. A 'Referral' button is highlighted with a red box. The 'Recording Summary' section provides key metrics: '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off', along with a date '2020-01-16 11:31', '256 Hz', and 'C3020195'. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section at the bottom shows an 'Order Holter report' button and a partially visible 'Ordered: 2025-02-12 10:41'.

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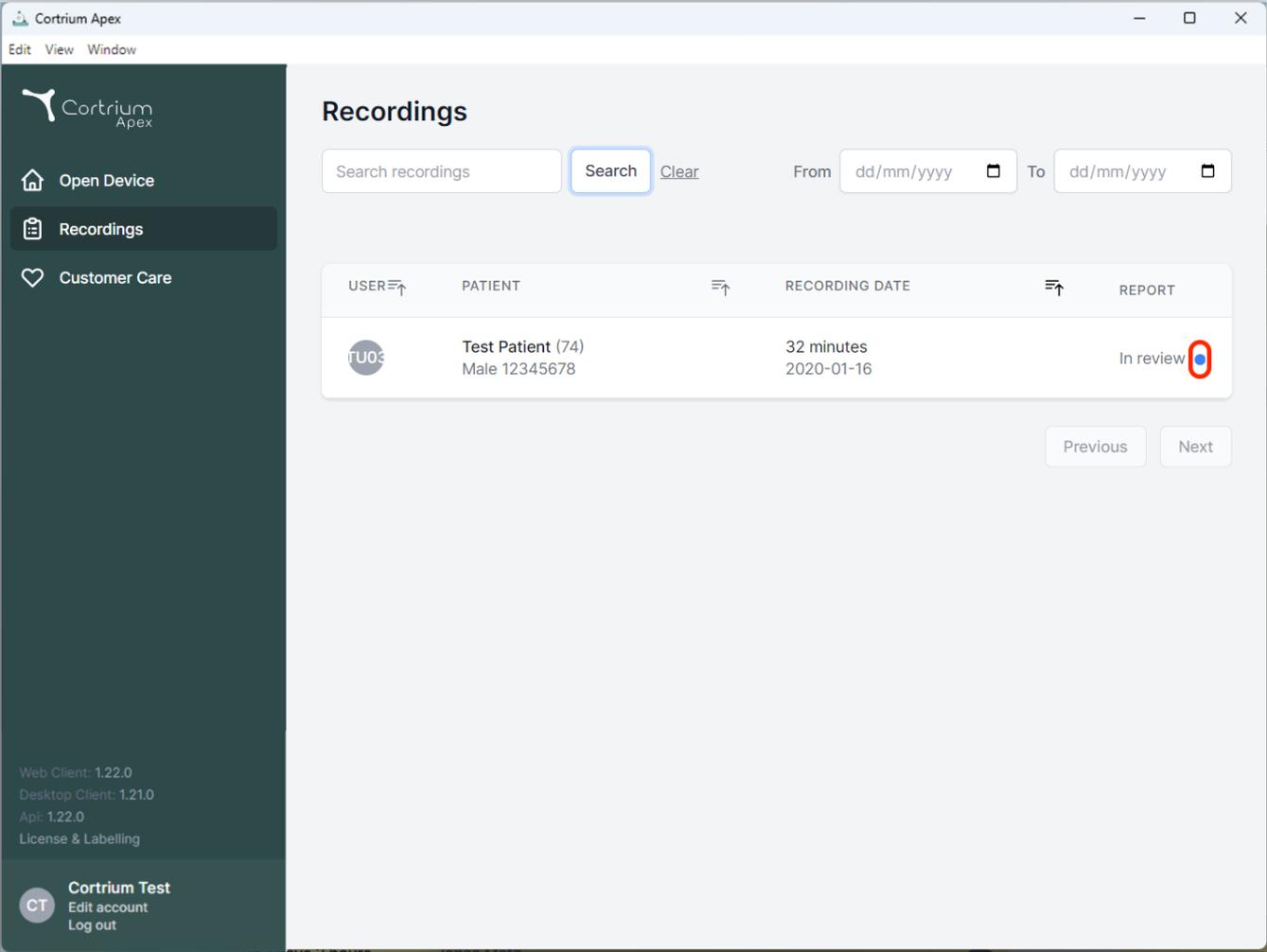
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a search bar with a 'Search' button (highlighted with a blue border) and a 'Clear' button. Below the search bar is a table with columns for USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for 'Test Patient (74)' with a recording date of '2020-01-16' and a duration of '32 minutes'. The status 'In review' is shown with a blue dot next to it. The sidebar on the left contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The bottom of the sidebar shows the user 'Cortrium Test' with options to 'Edit account' and 'Log out'.

| USER | PATIENT | RECORDING DATE | REPORT |
|------|------------------------------------|--------------------------|-----------|
| TU03 | Test Patient (74) Male 12345678 | 32 minutes 2020-01-16 | In review |

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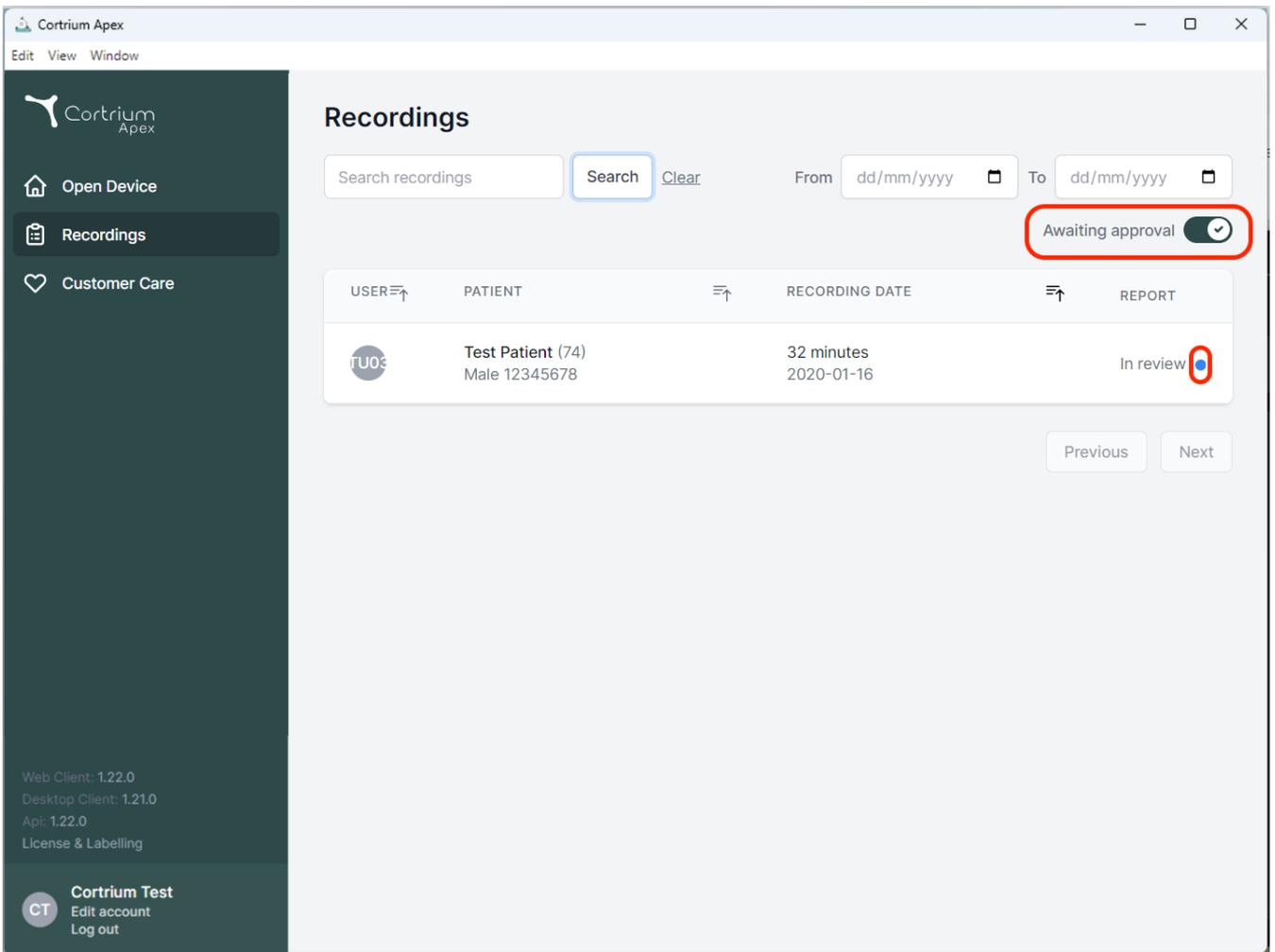
The screenshot displays the 'Recordings' section of the Cortrium Apex application. The interface includes a search bar, a 'Search' button, and a 'Clear' link. There are date pickers for 'From' and 'To' filters. A toggle switch labeled 'Awaiting approval' is highlighted with a red box and is currently turned on. Below this, a table lists recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. The first row shows a recording for 'Test Patient (74)' with a recording date of '2020-01-16' and a duration of '32 minutes'. The 'REPORT' column for this row shows 'In review' with a blue dot next to it, also highlighted with a red box. At the bottom of the table, there are 'Previous' and 'Next' navigation buttons. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The bottom of the sidebar shows version information and user options: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', 'License & Labelling', and 'Cortrium Test' with 'Edit account' and 'Log out' options.

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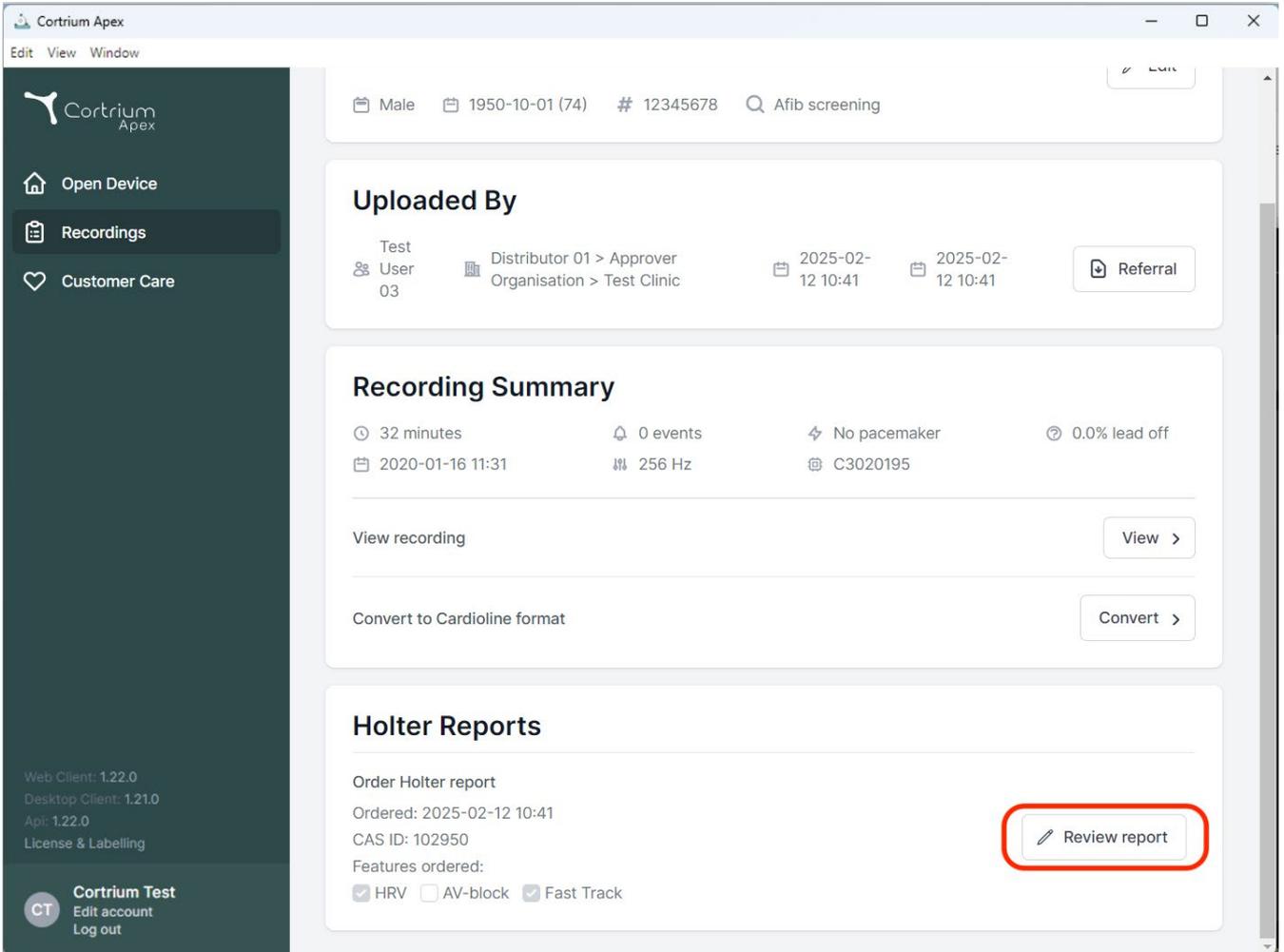
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes "Edit", "View", and "Window" options. The left sidebar contains navigation links for "Open Device", "Recordings", and "Customer Care". The main content area is divided into several sections:

- Patient Information:** Male, 1950-10-01 (74), # 12345678, Afib screening.
- Uploaded By:** Test User 03, Distributor 01 > Approver Organisation > Test Clinic, 2025-02-12 10:41, 2025-02-12 10:41, Referral button.
- Recording Summary:** 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, C3020195. Includes "View recording" and "Convert to Cardioline format" buttons.
- Holter Reports:** Order Holter report, Ordered: 2025-02-12 10:41, CAS ID: 102950, Features ordered: HRV, AV-block, Fast Track. A "Review report" button is highlighted with a red box.

Footer information includes: Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling, and user profile for Cortrium Test (Edit account, Log out).

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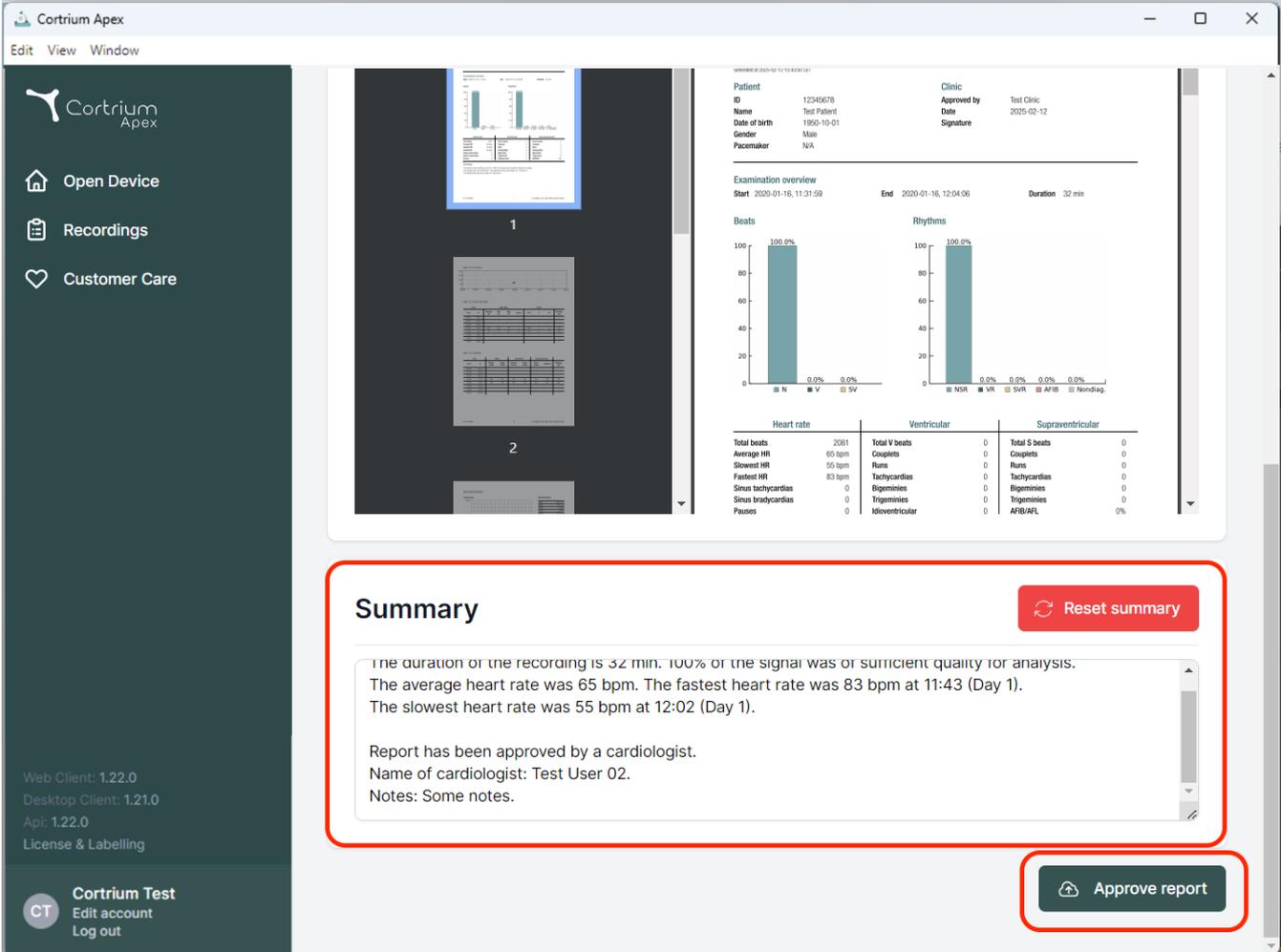The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Header:** 'Cortrium Apex' logo and window controls.
- Patient Information:** ID 12345678, Name Test Patient, Date of birth 1950-10-01, Gender Male, Pacemaker N/A. Clinic: Test Clinic, Approved by Test Clinic, Date 2025-02-12.
- Examination overview:** Start 2020-01-16, 11:31:59; End 2020-01-16, 12:04:06; Duration 32 min.
- Beats and Rhythms:** Two bar charts showing 100.0% for 'N' (Normal Sinus Rhythm) and 0.0% for other categories (V, SV, NSR, VR, SVR, AFIB, Nondiag).
- Summary Table:**

| Heart rate | | Ventricular | | Supraventricular | |
|--------------------|--------|-----------------|---|------------------|----|
| Total beats | 2081 | Total V beats | 0 | Total S beats | 0 |
| Average HR | 65 bpm | Couplets | 0 | Couplets | 0 |
| Slowest HR | 55 bpm | Runs | 0 | Runs | 0 |
| Fastest HR | 83 bpm | Tachycardias | 0 | Tachycardias | 0 |
| Sinus tachycardias | 0 | Bigeminy | 0 | Bigeminy | 0 |
| Sinus bradycardias | 0 | Trigeminy | 0 | Trigeminy | 0 |
| Pauses | 0 | Idioventricular | 0 | AFIB/AFL | 0% |
- Summary Section (highlighted with a red box):**

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.
- Approval Section (highlighted with a red box):** Approve report

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033