Cortrium Apex 1.22

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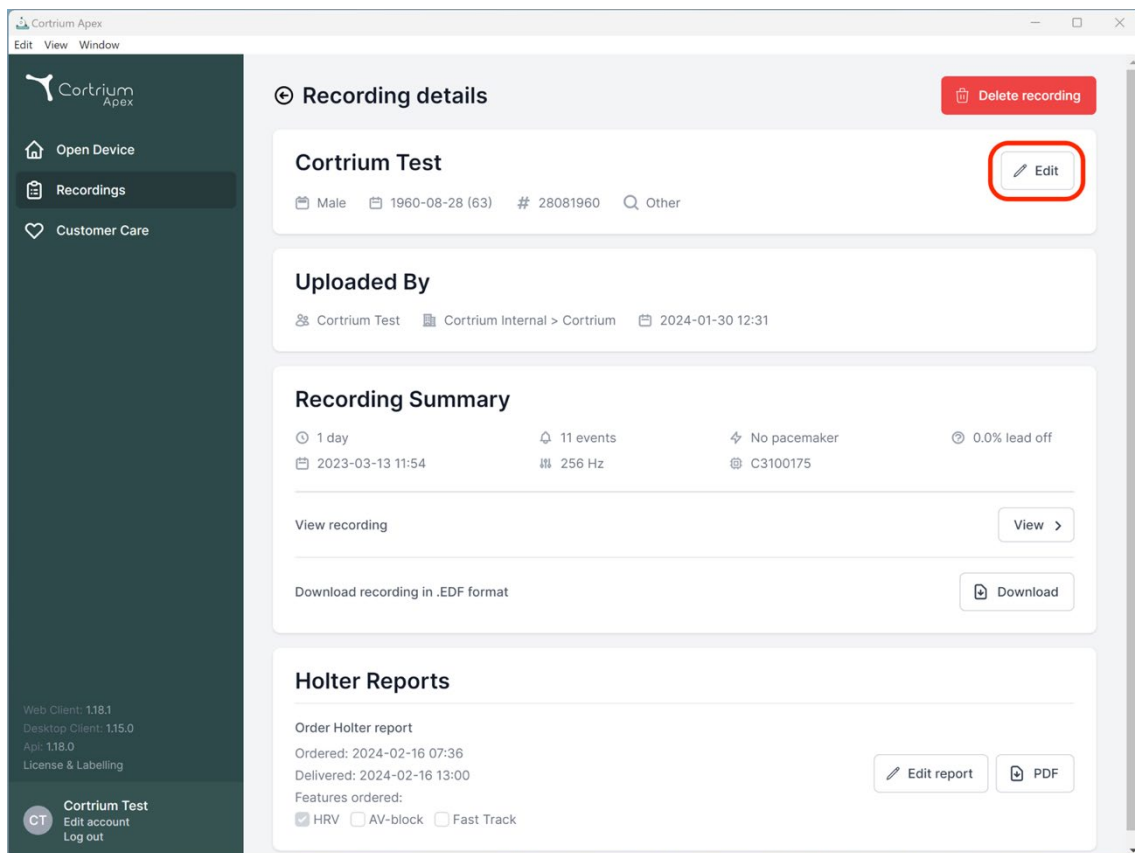
Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary
 1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports
 Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
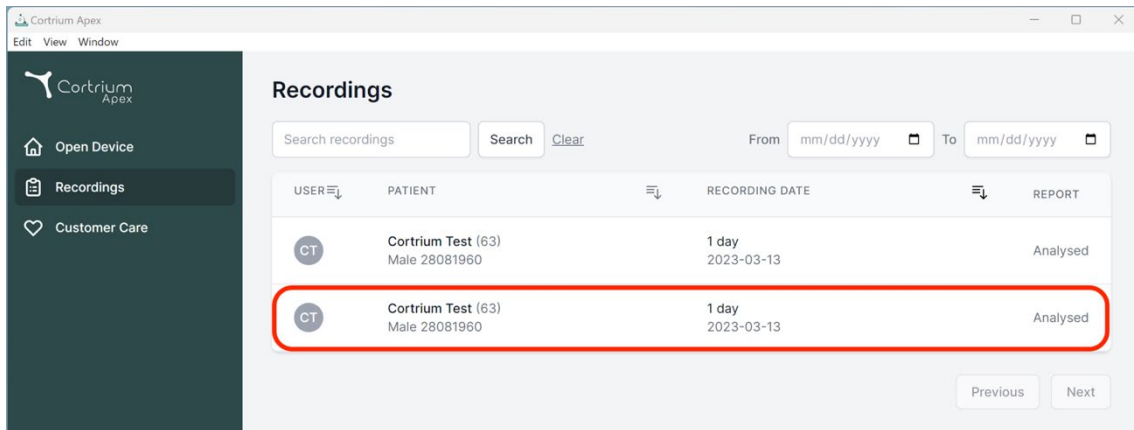
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

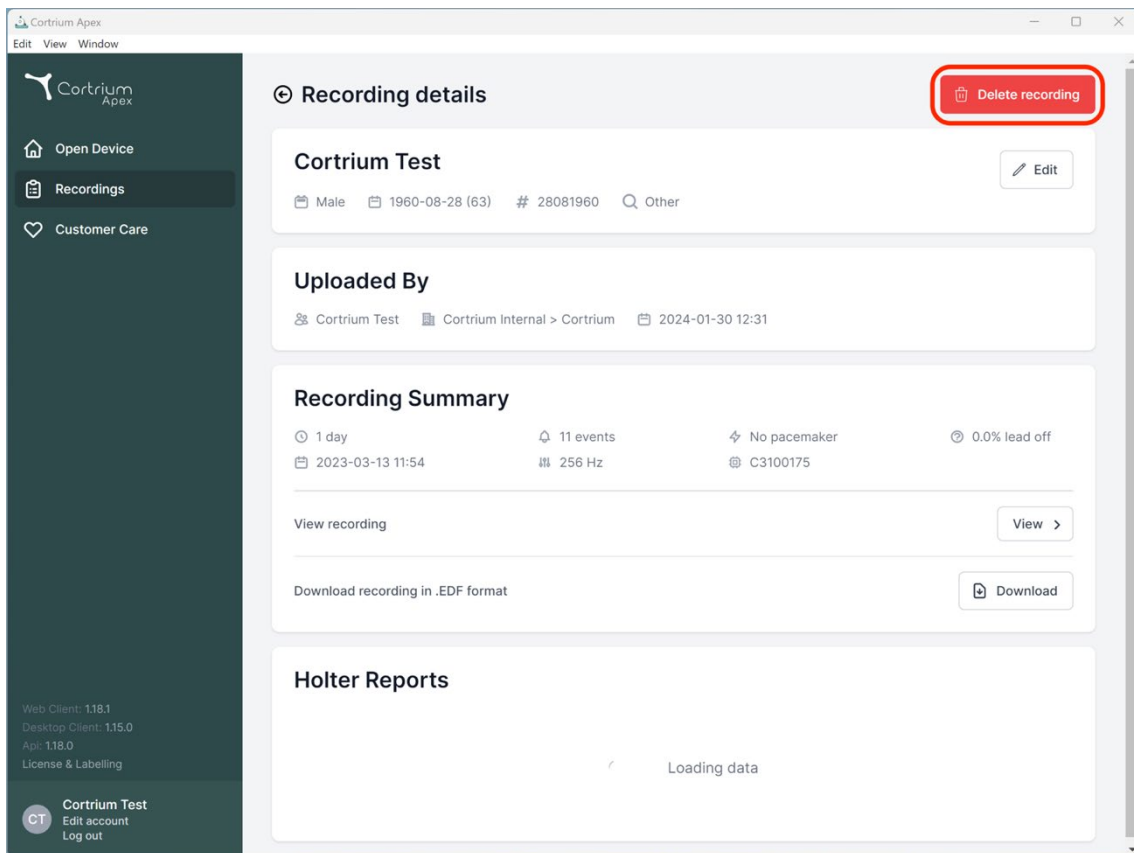Previous Next

Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View](#)

Download recording in .EDF format [Download](#)

Holter Reports

Loading data

Web Client: 1.18.1
Desktop Client: 1.15.0
API: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

Cortrium Test 

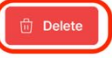
 Male  1960-08-28 (63)  # 28081960  Other


Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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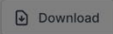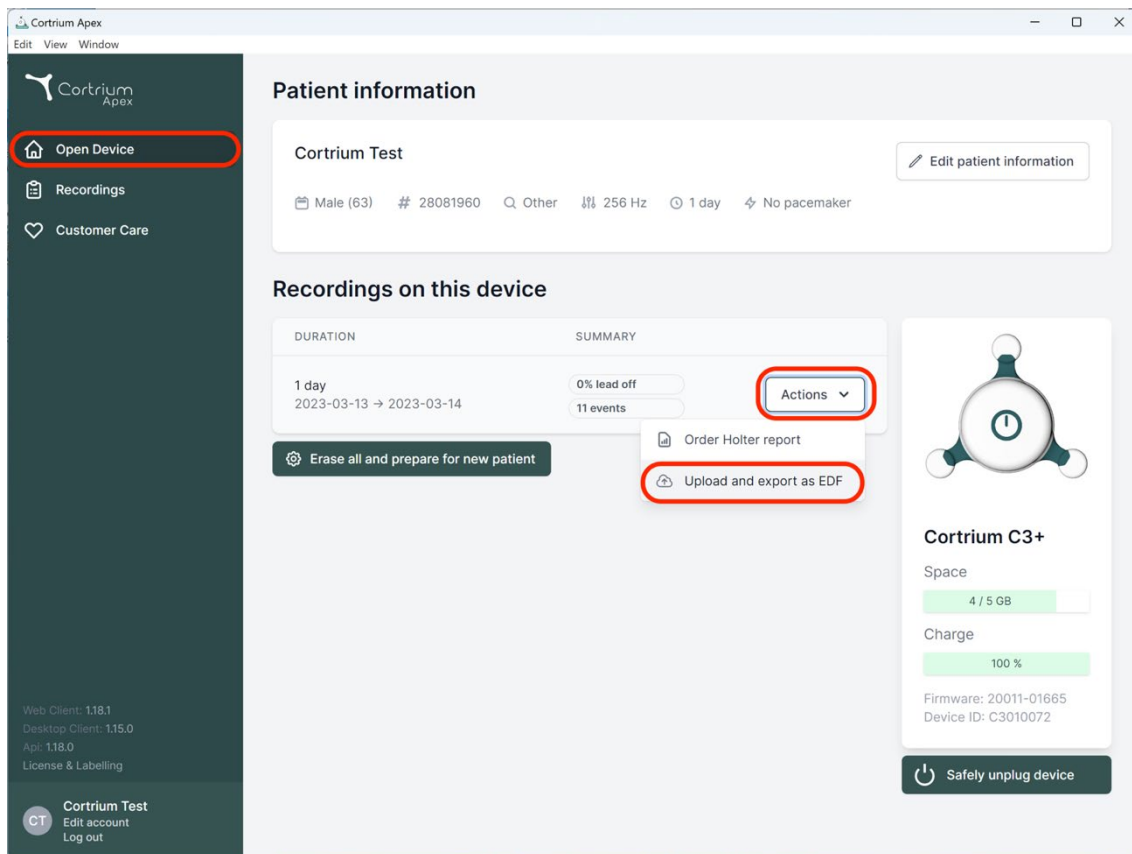
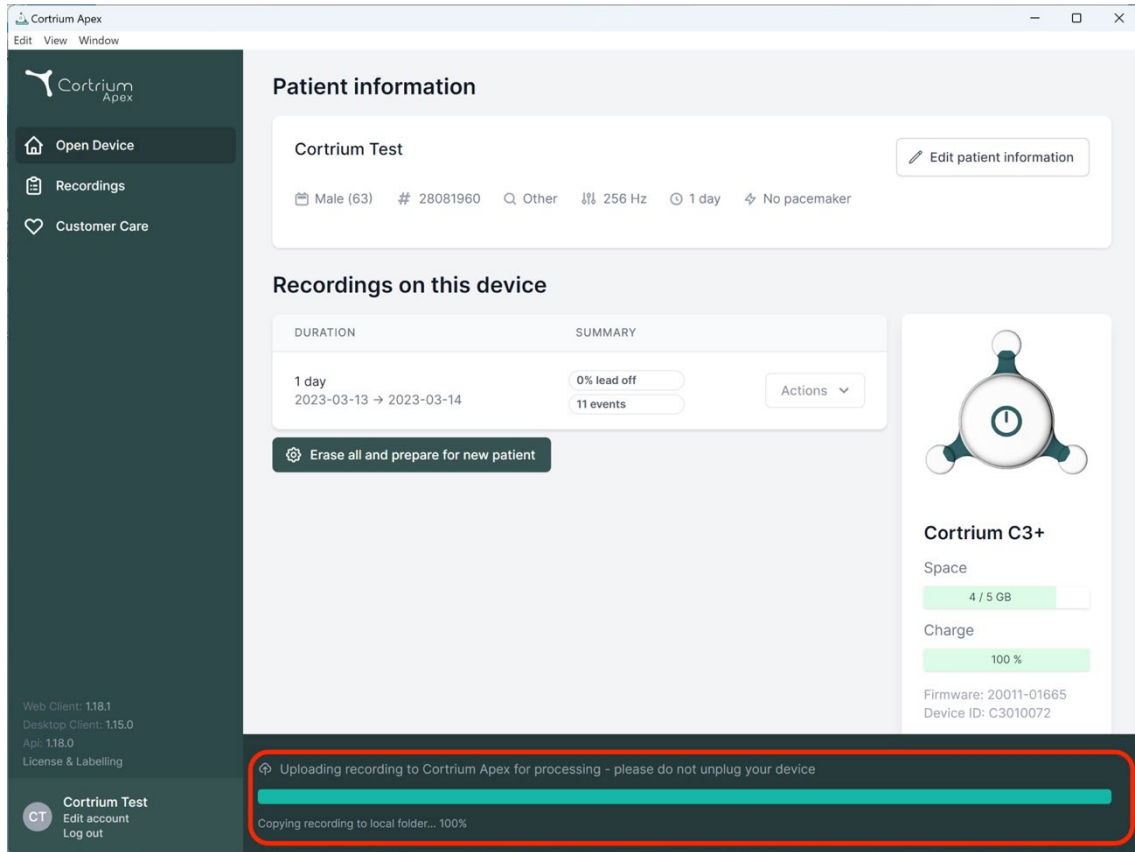
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

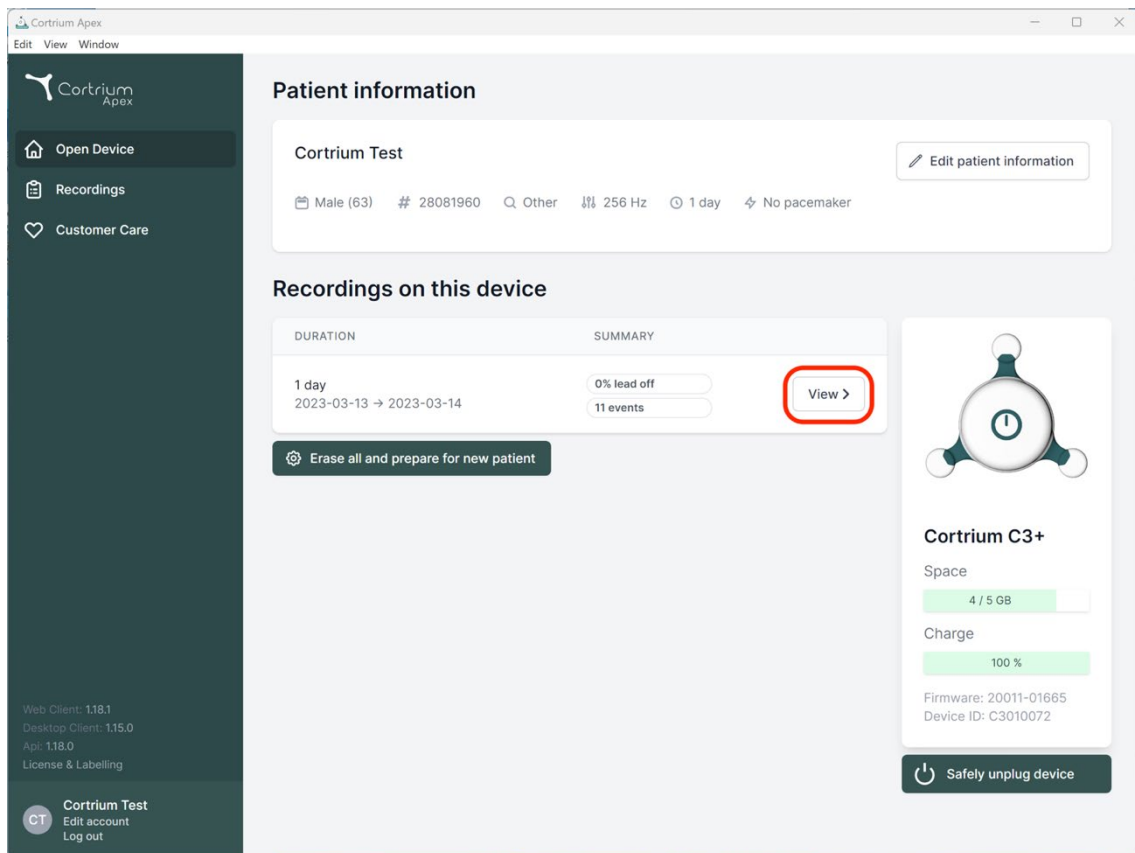
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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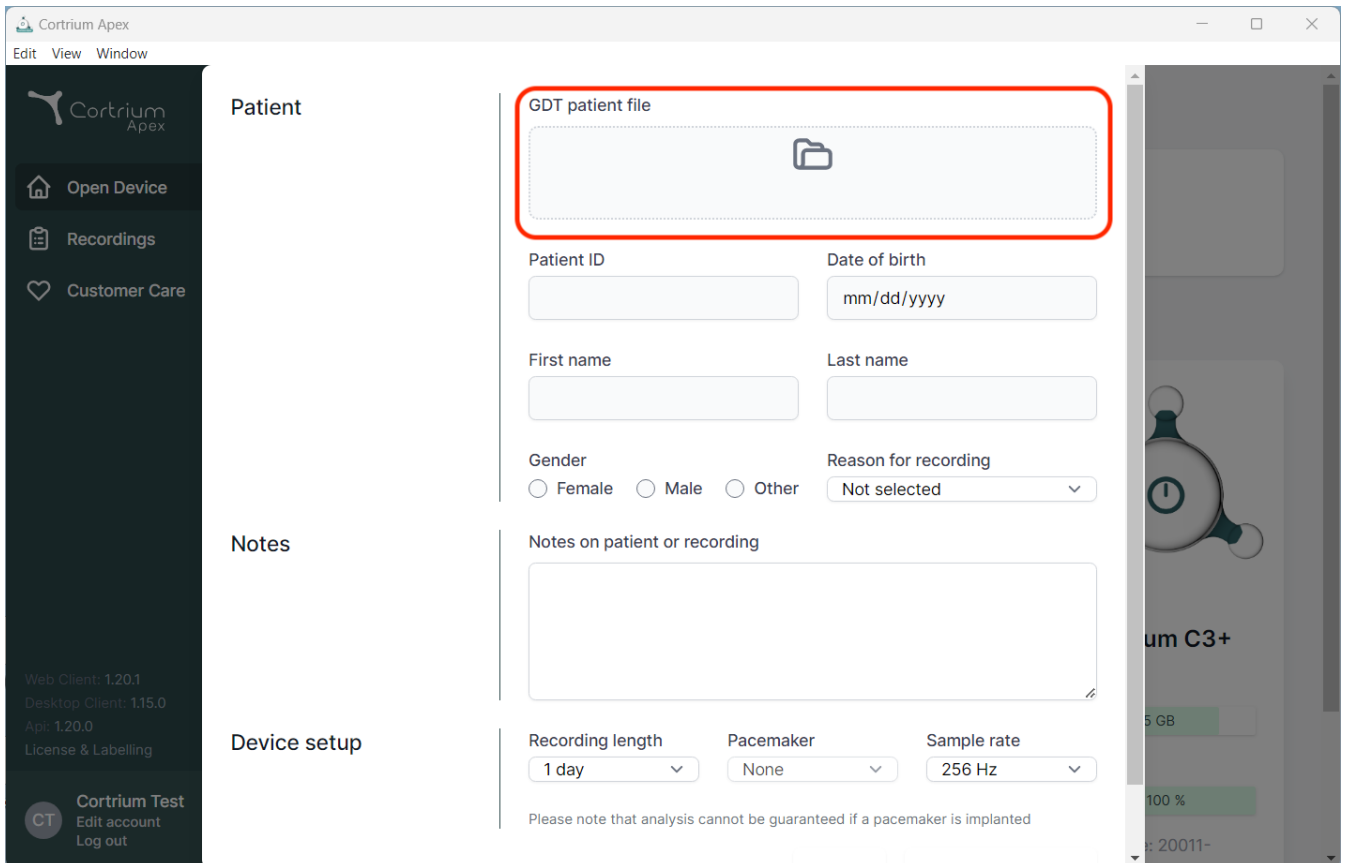
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

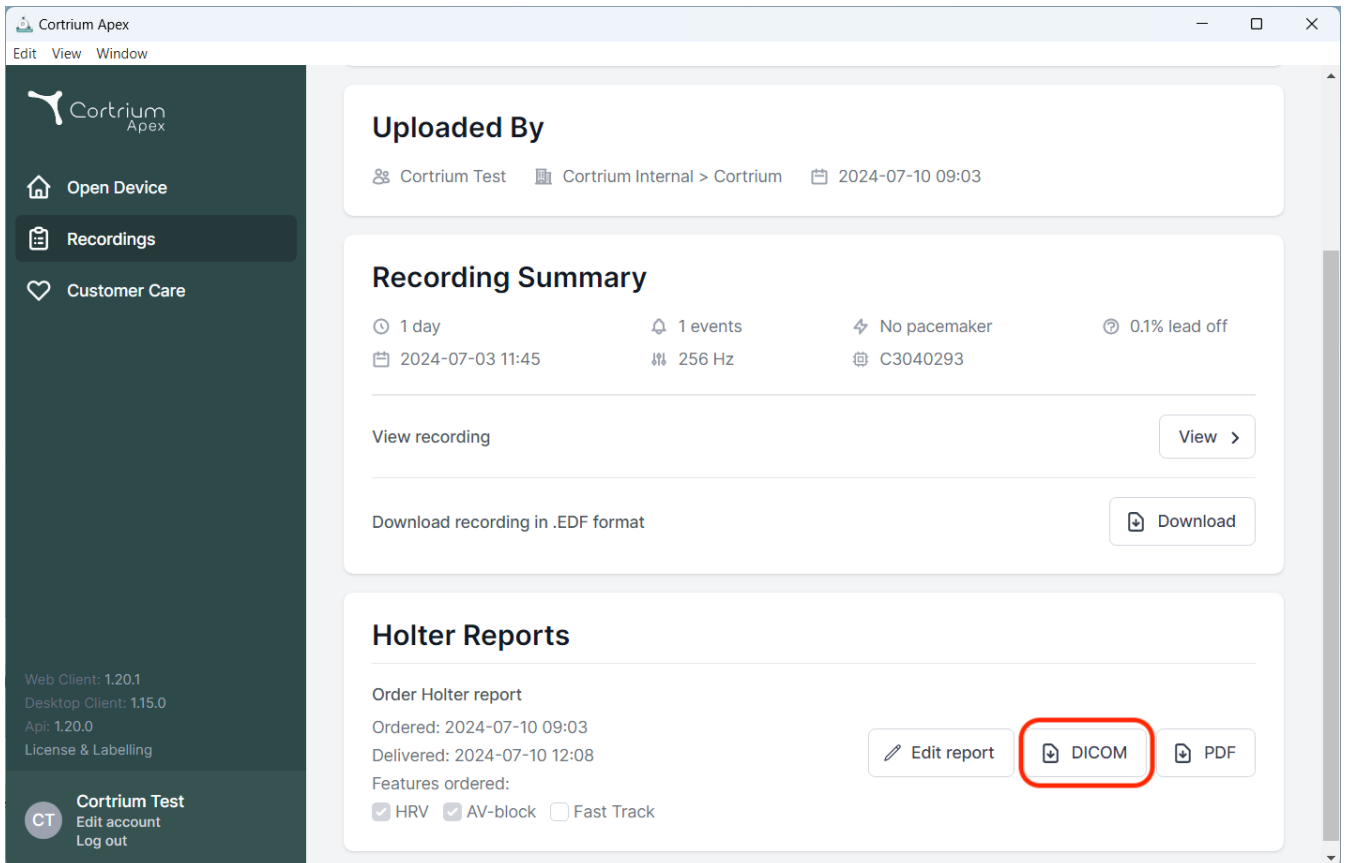
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section has a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). Below these is a disclaimer: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device model, "um C3+", is visible, showing a battery level of "100 %" and a storage capacity of "5 GB".

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
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details including '1 day', '1 events', 'No pacemaker', and '0.1% lead off'. It includes a 'View recording' button and a 'Download recording in .EDF format' button.
- Holter Reports:** Shows the 'Order Holter report' section with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. The 'DICOM' button is highlighted with a red rectangle.

The bottom of the sidebar shows the user 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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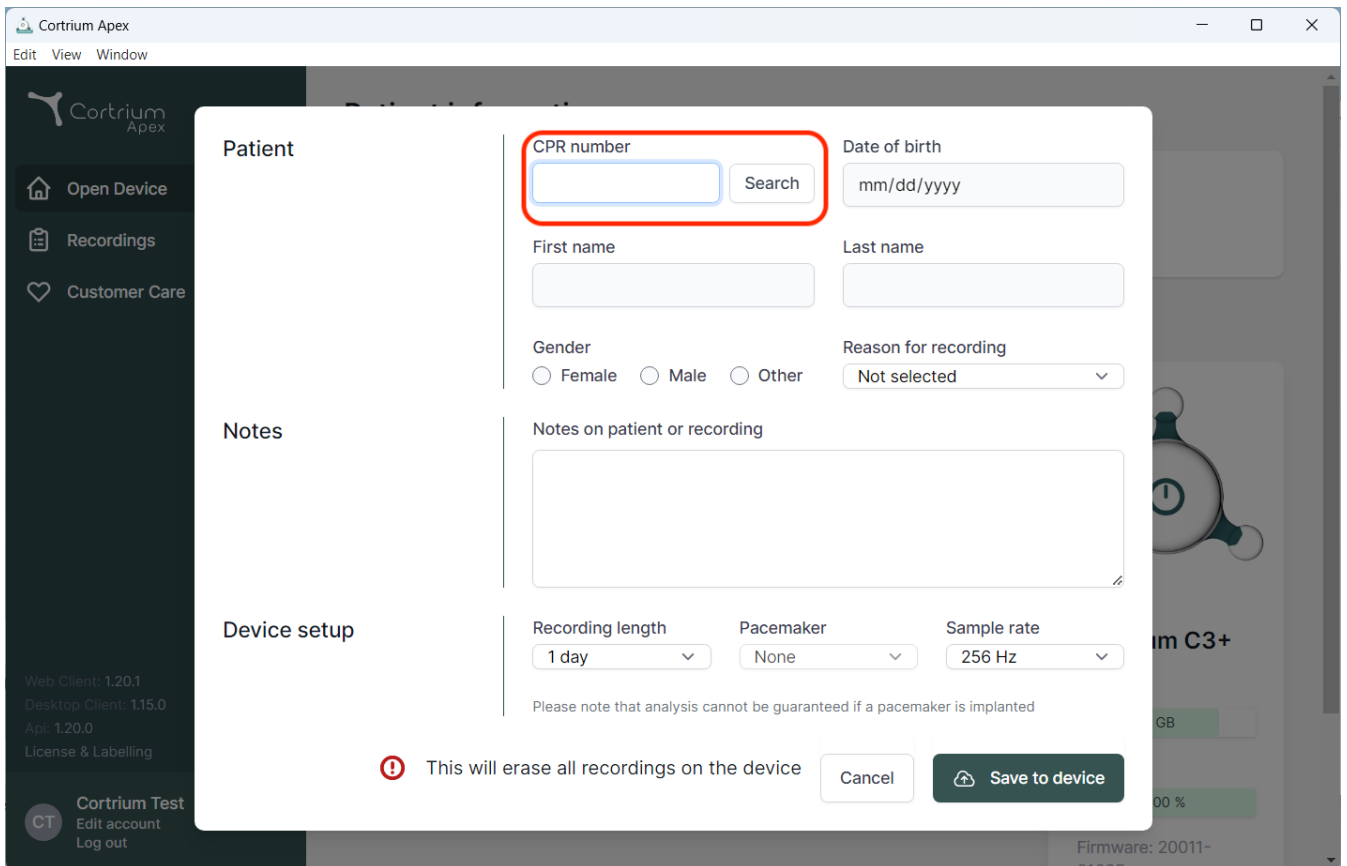The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for searching and entering patient information. The form includes fields for CPR number, Date of birth, First name, Last name, Gender, Reason for recording, and a large text area for Notes. The CPR number field is highlighted with a red rectangle. Below the form, there is a "Device setup" section with dropdown menus for Recording length, Pacemaker, and Sample rate. A warning message at the bottom states: "This will erase all recordings on the device". The background shows the main application menu with options like "Open Device", "Recordings", and "Customer Care".

Patient

CPR number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

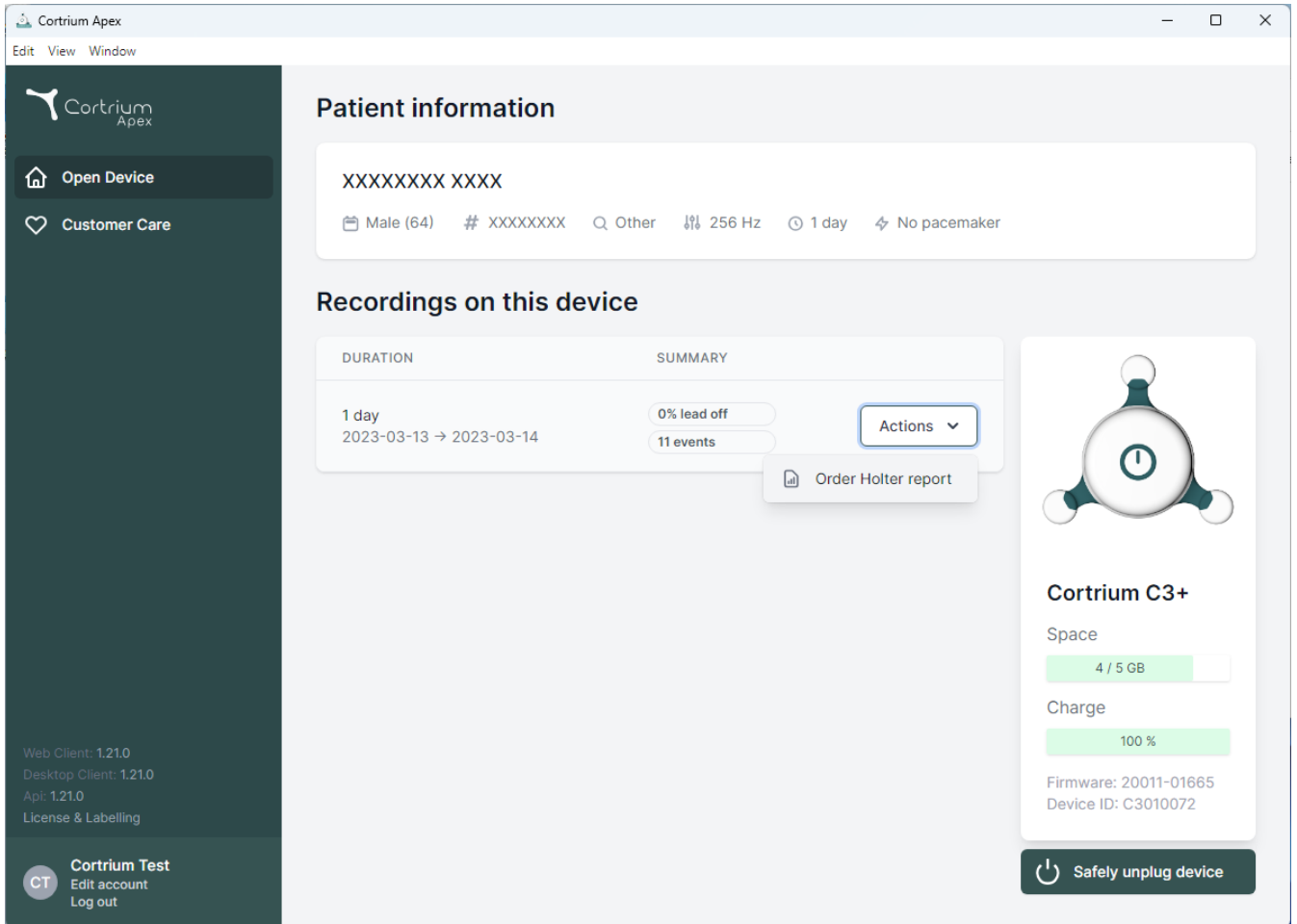
Recording length Pacemaker Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device' and 'Customer Care'. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions

[Order Holter report](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

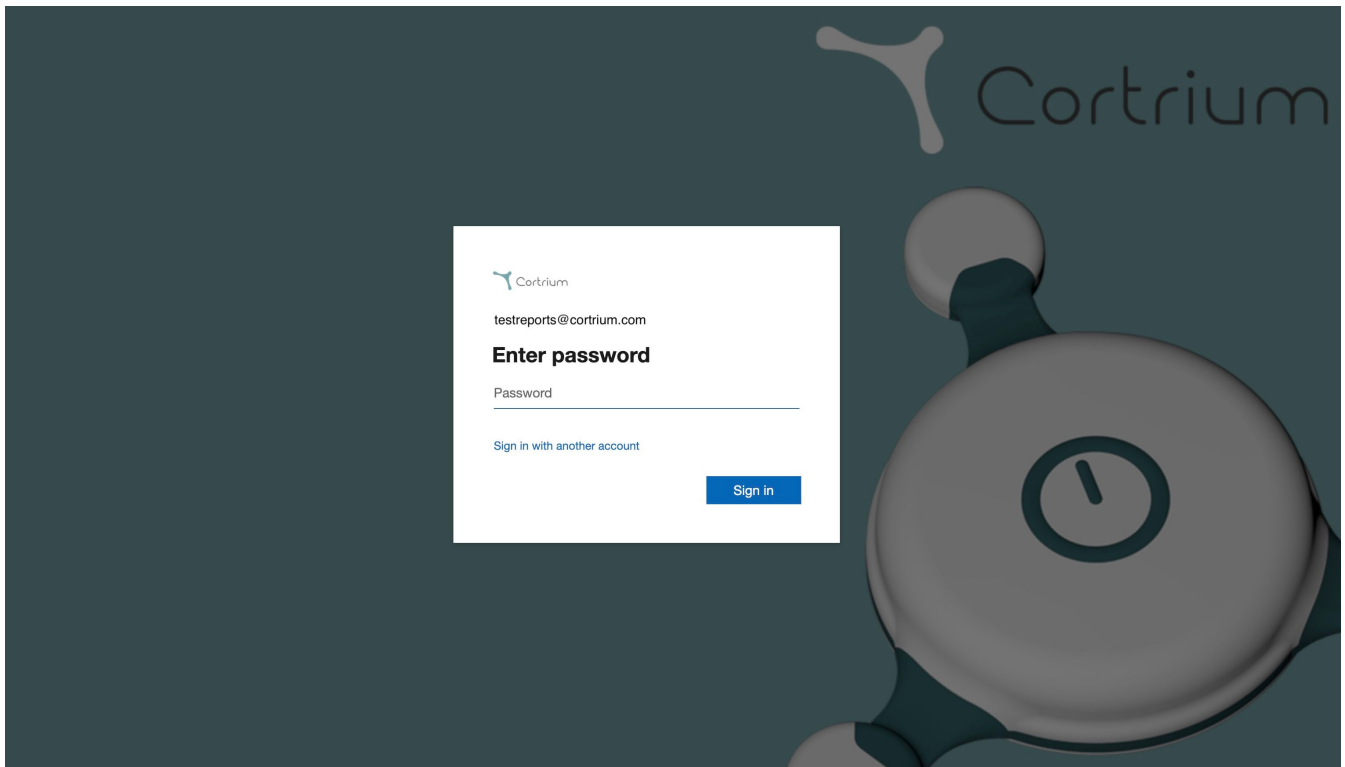
[Safely unplug device](#)

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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
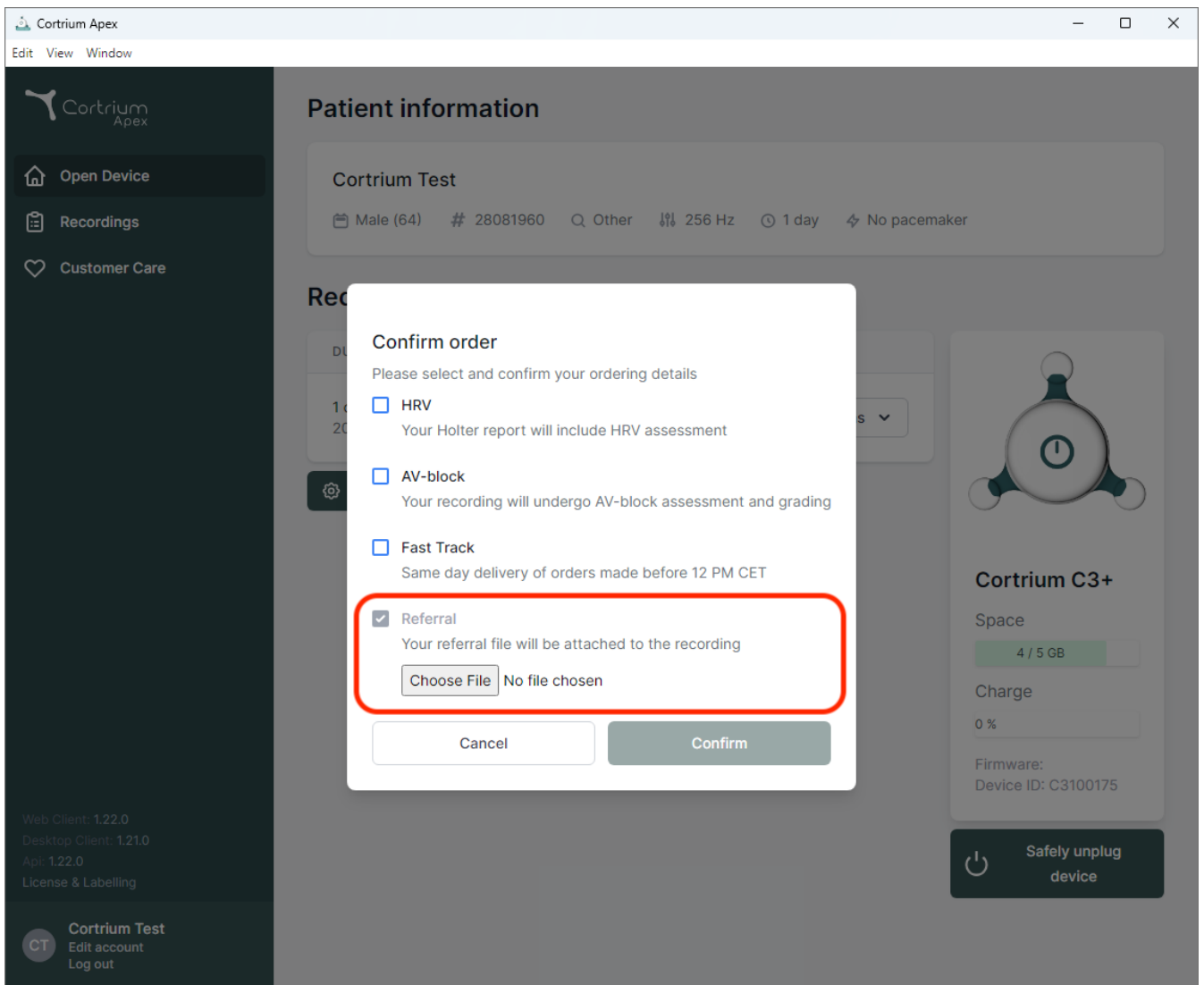
The screenshot shows the Cortrium Apex web application interface. A 'Confirm order' dialog box is displayed in the center, overlaid on the 'Patient information' page. The dialog box contains the following options:

- ☐ HRV
Your Holter report will include HRV assessment
- ☐ AV-block
Your recording will undergo AV-block assessment and grading
- ☐ Fast Track
Same day delivery of orders made before 12 PM CET
- ☒ Referral
Your referral file will be attached to the recording

The 'Referral' option is highlighted with a red rectangle. Below the 'Referral' option, there is a 'Choose File' button and the text 'No file chosen'. At the bottom of the dialog box, there are 'Cancel' and 'Confirm' buttons.

The background interface shows the 'Patient information' section with details for 'Cortrium Test' (Male (64), # 28081960, Q Other, 256 Hz, 1 day, No pacemaker). On the right, there is a section for 'Cortrium C3+' showing 'Space' (4 / 5 GB), 'Charge' (0 %), and 'Firmware: Device ID: C3100175'. A 'Safely unplug device' button is visible at the bottom right.

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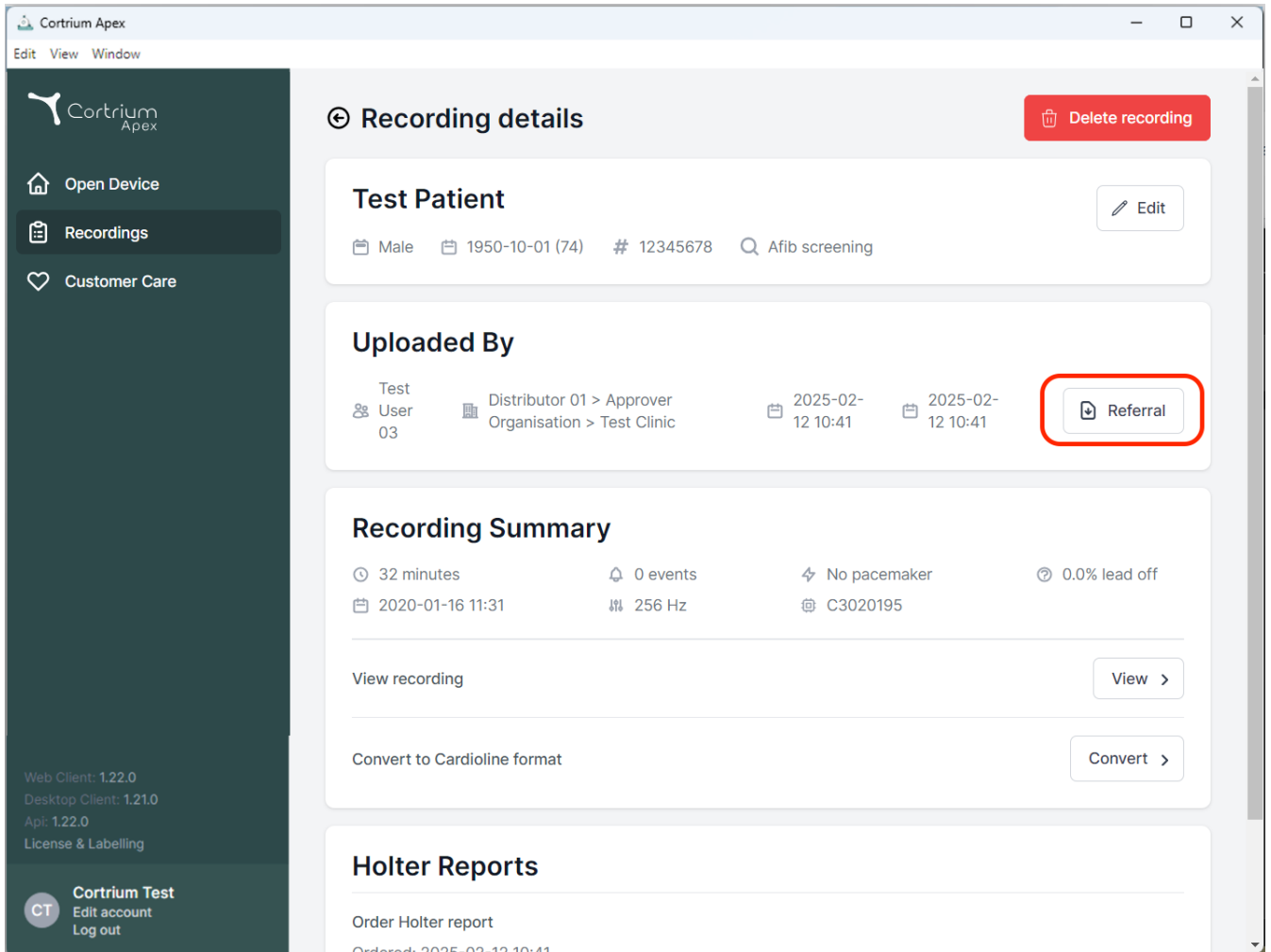The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right corner.

Test Patient section includes patient information: Male, 1950-10-01 (74), ID 12345678, and Afib screening. An 'Edit' button is available.

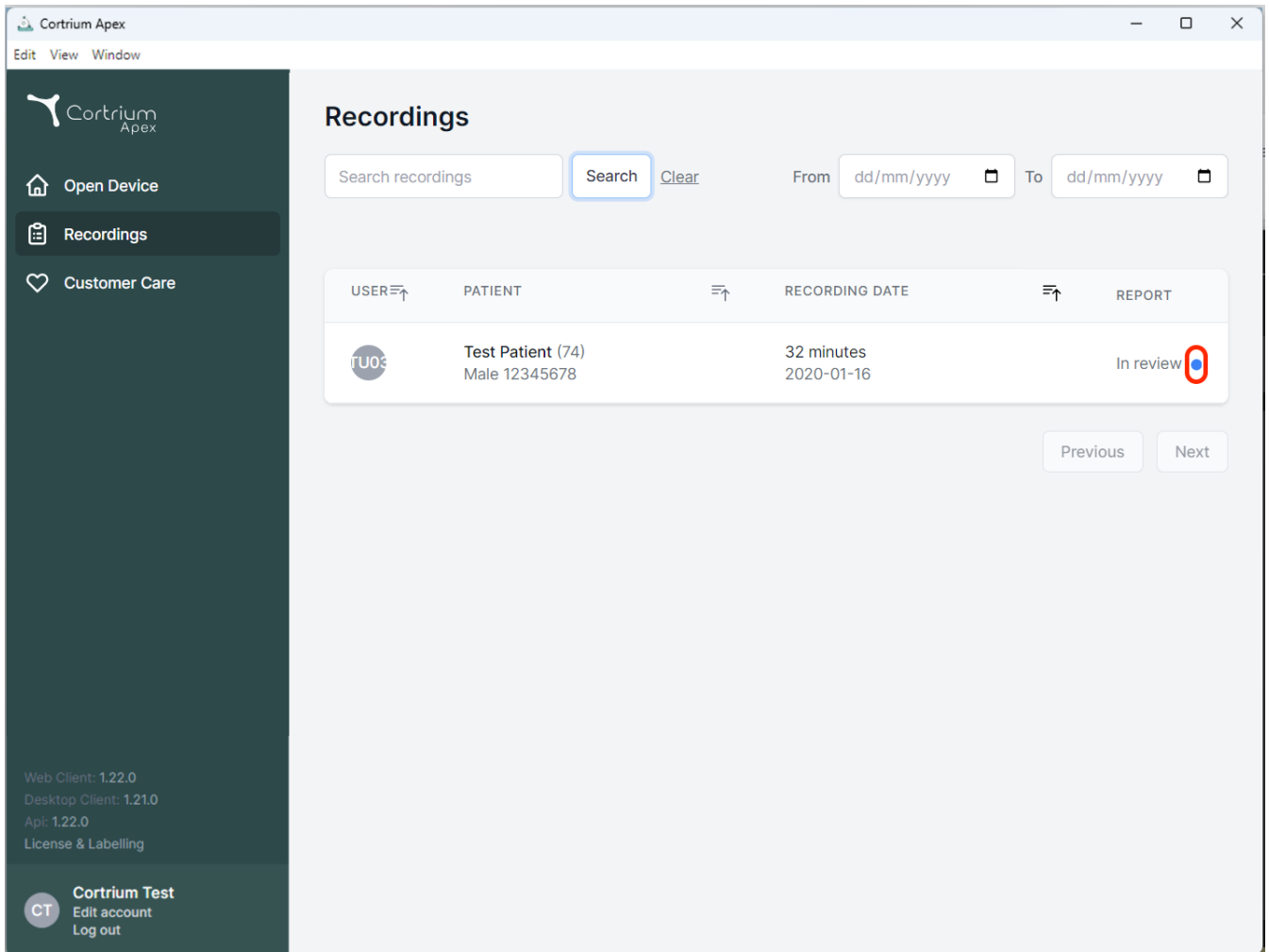
Uploaded By section shows the user 'User 03' and the distributor 'Distributor 01 > Approver Organisation > Test Clinic'. It also displays the upload date and time '2025-02-12 10:41'. A 'Referral' button is highlighted with a red box.

Recording Summary section provides key metrics: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date '2020-01-16 11:31', frequency '256 Hz', and device ID 'C3020195'. Below this, there are buttons for 'View recording' and 'Convert to Cardioline format'.

Holter Reports section includes an 'Order Holter report' button and a note 'Ordered: 2025-02-12 10:41'.

The sidebar at the bottom left contains version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling) and user options for 'Cortrium Test' (Edit account, Log out).

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
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation options: Open Device, Recordings (selected), and Customer Care. The main content area is titled "Recordings" and features a search bar with a "Search" button (highlighted with a blue border) and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for "Test Patient (74)" with a recording date of "2020-01-16" and a duration of "32 minutes". The "REPORT" column shows "In review" with a blue dot icon circled in red. At the bottom of the sidebar, there is a "Cortrium Test" section with a user profile icon and links for "Edit account" and "Log out".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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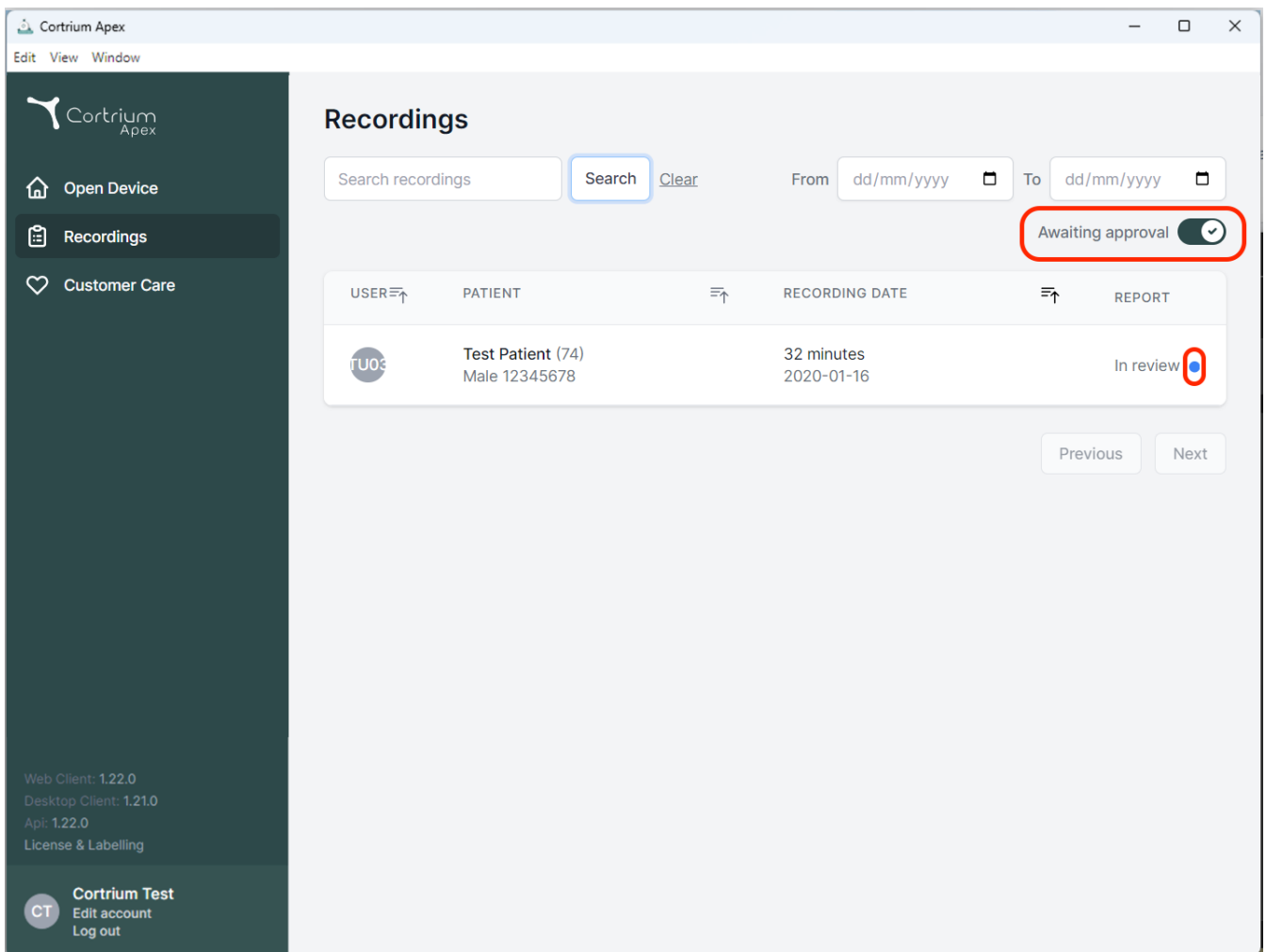
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings (selected), and Customer Care. The main content area is titled "Recordings" and includes a search bar, a "Search" button, a "Clear" link, and date filters for "From" and "To". A toggle switch labeled "Awaiting approval" is set to "On" and is highlighted with a red box. Below this is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a duration of "32 minutes". The "REPORT" column for this entry shows "In review" with a blue dot icon, which is also highlighted with a red box. At the bottom of the table are "Previous" and "Next" buttons. The footer of the sidebar displays version information: Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling, and a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

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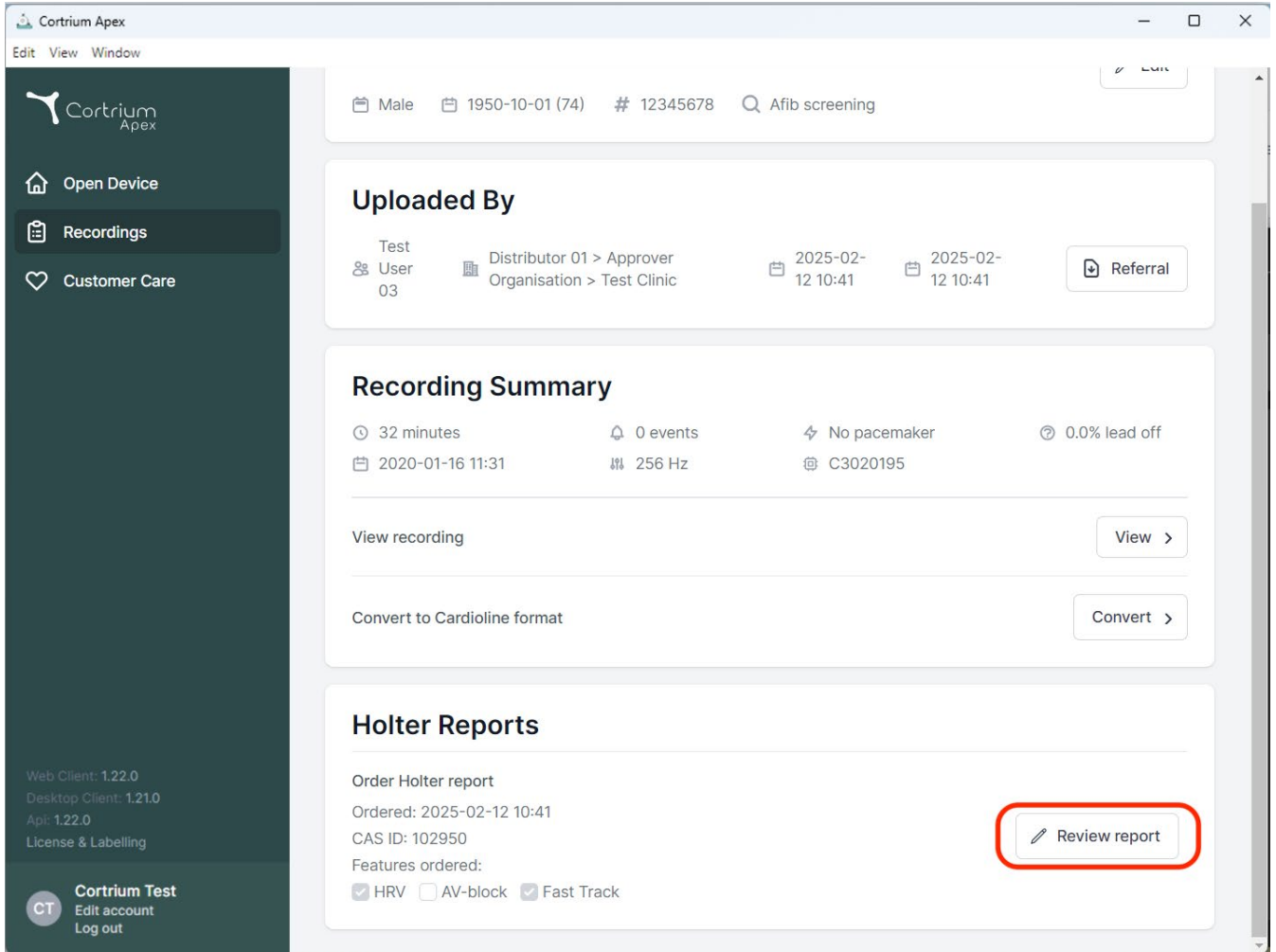The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient details: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41 and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

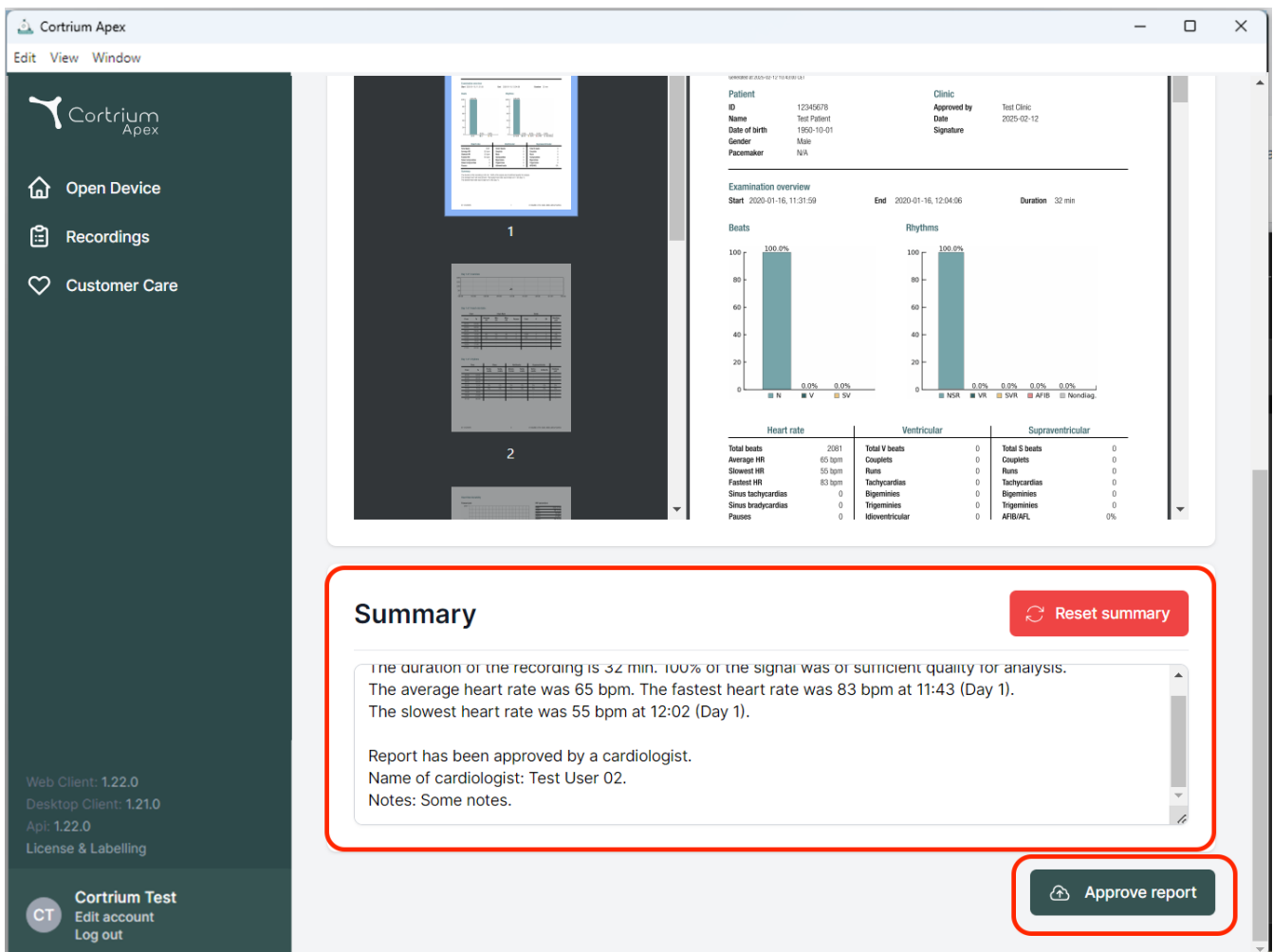
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Apl: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

Beats
100.0%
0.0% 0.0% 0.0%

Rhythms
100.0%
0.0% 0.0% 0.0% 0.0% 0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigemines	0	Bigemines	0
Sinus bradycardias	0	Trigemines	0	Trigemines	0
Pauses	0	Idioventricular	0	AF/IB/AFI	0%

Summary [Reset summary](#)

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

[Approve report](#)

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denemarken
E-mail: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033