Cortrium Apex 1.22

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
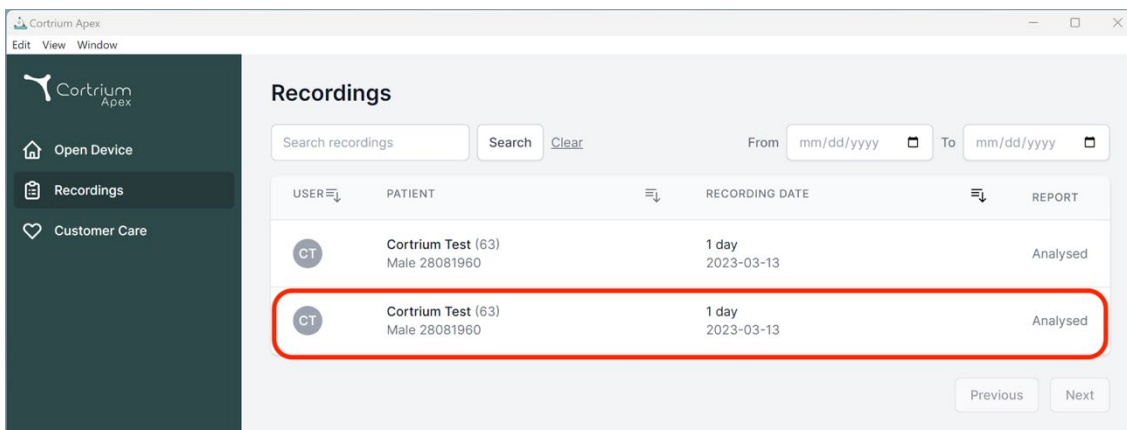
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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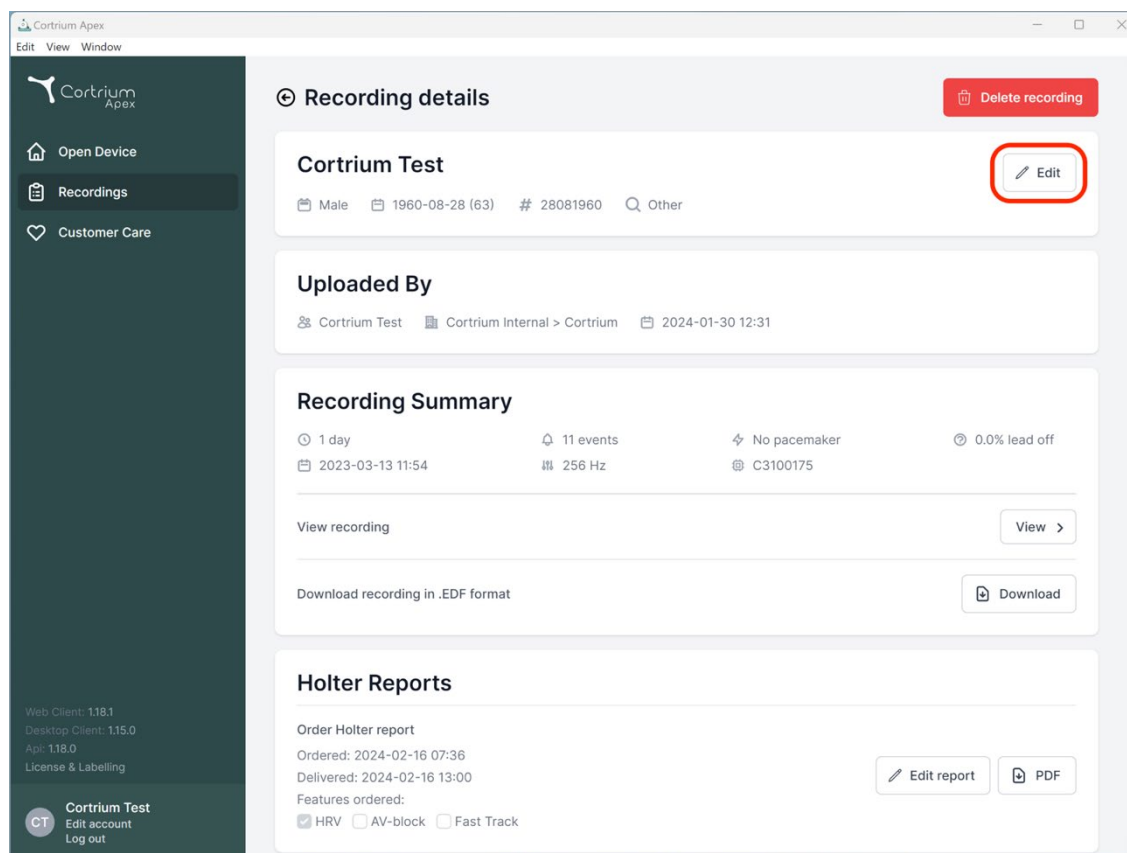2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
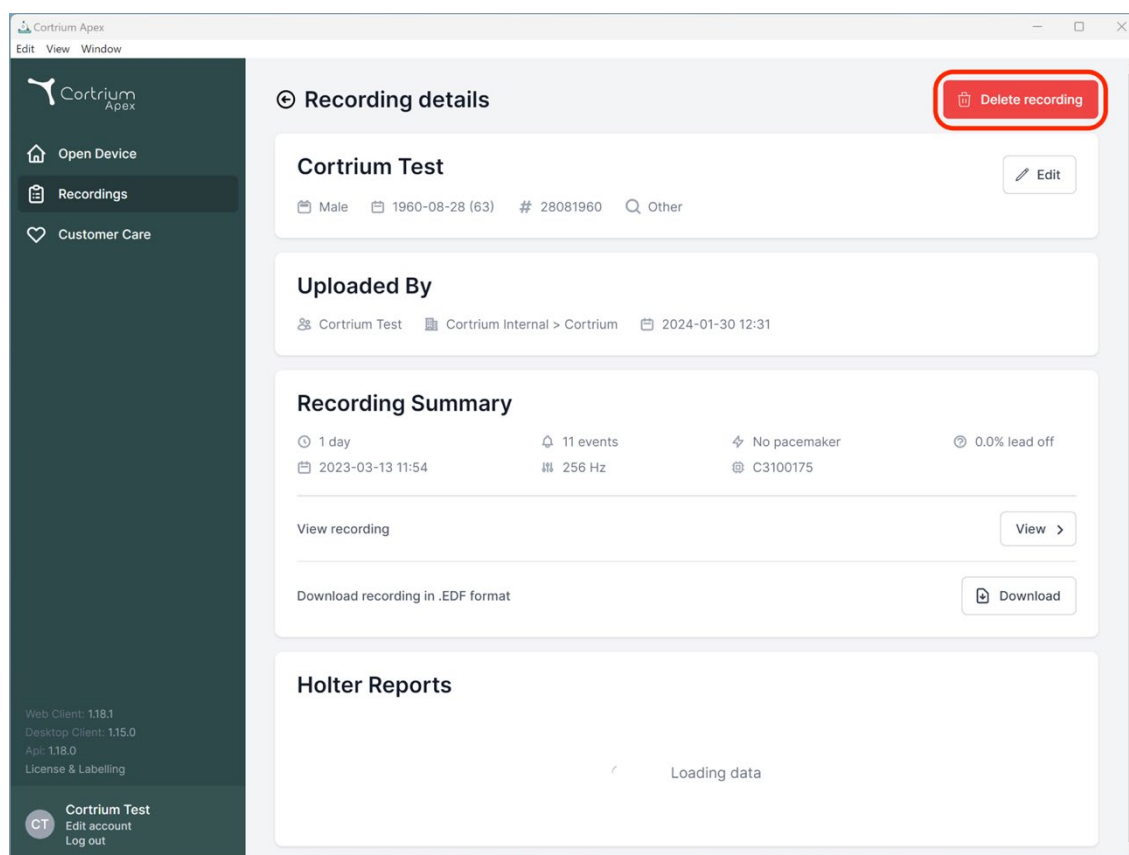
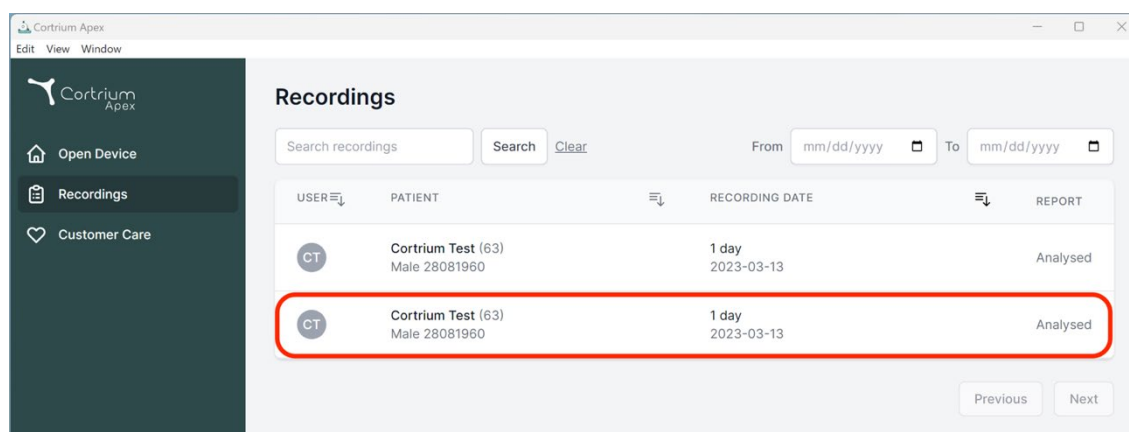
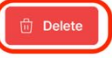
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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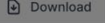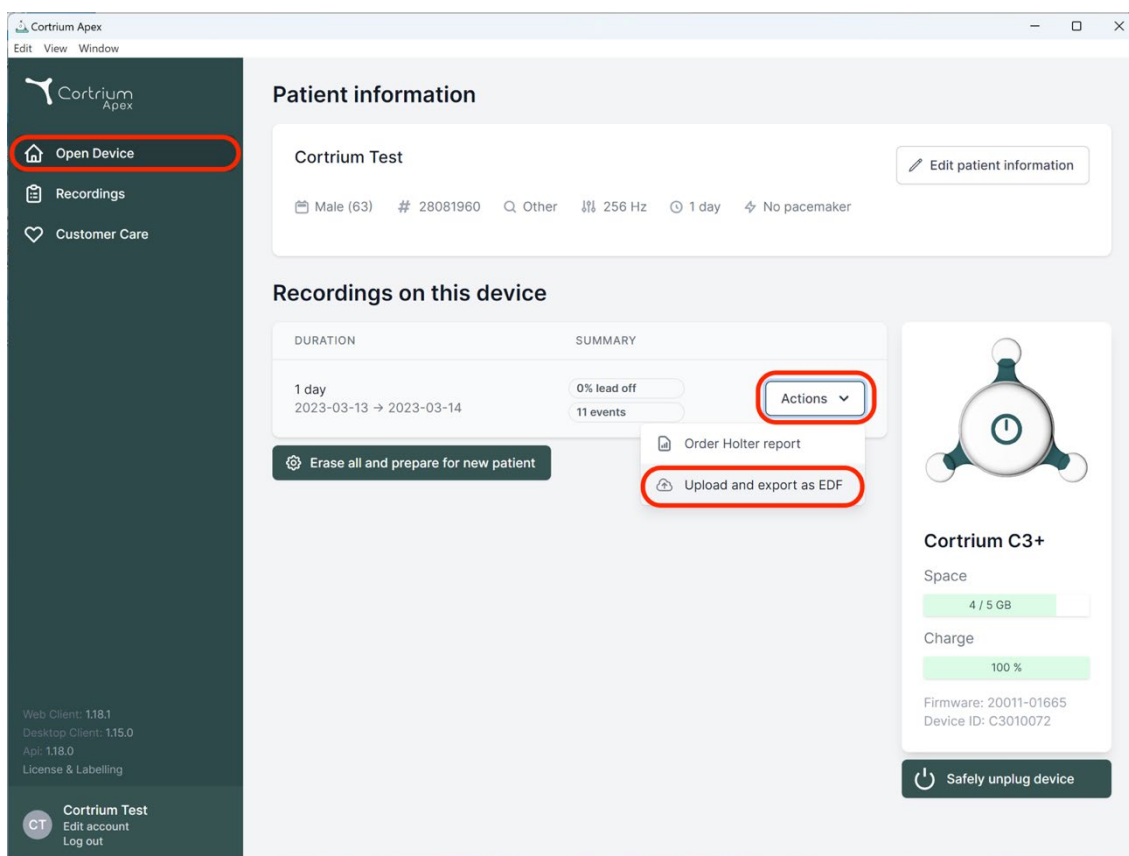
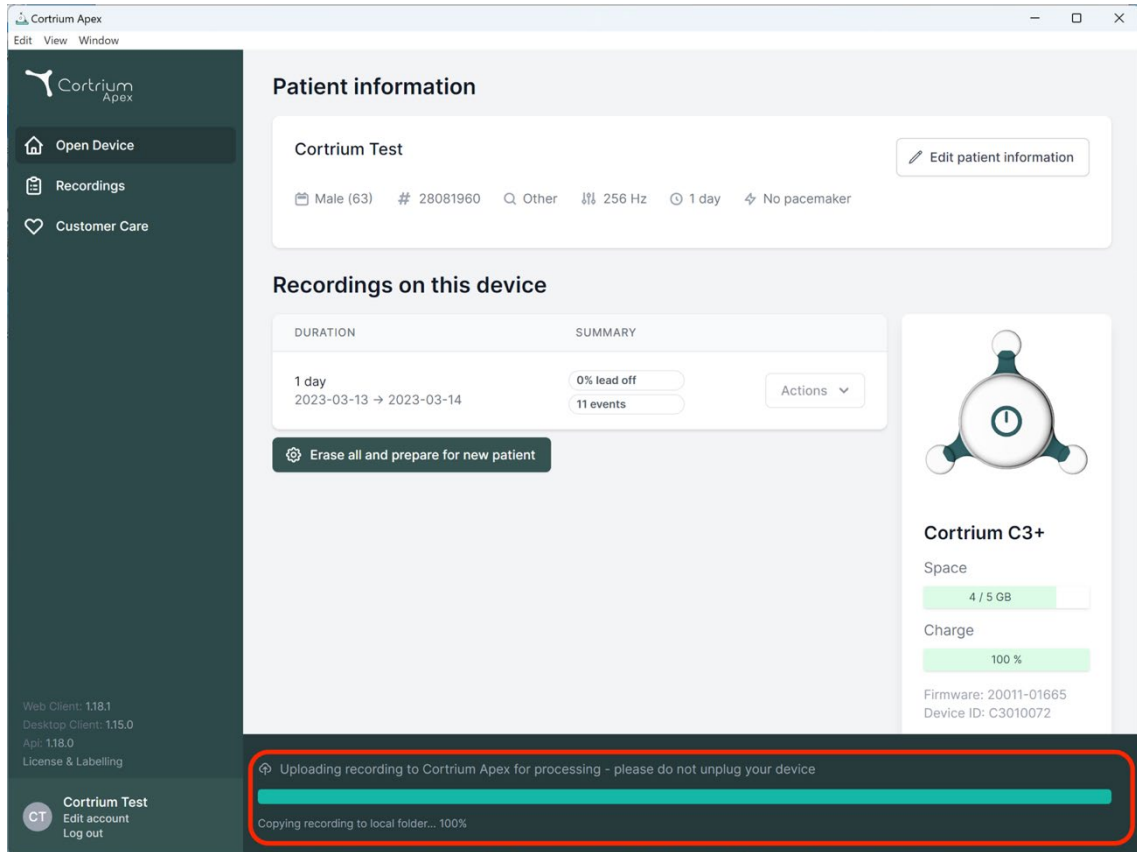
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

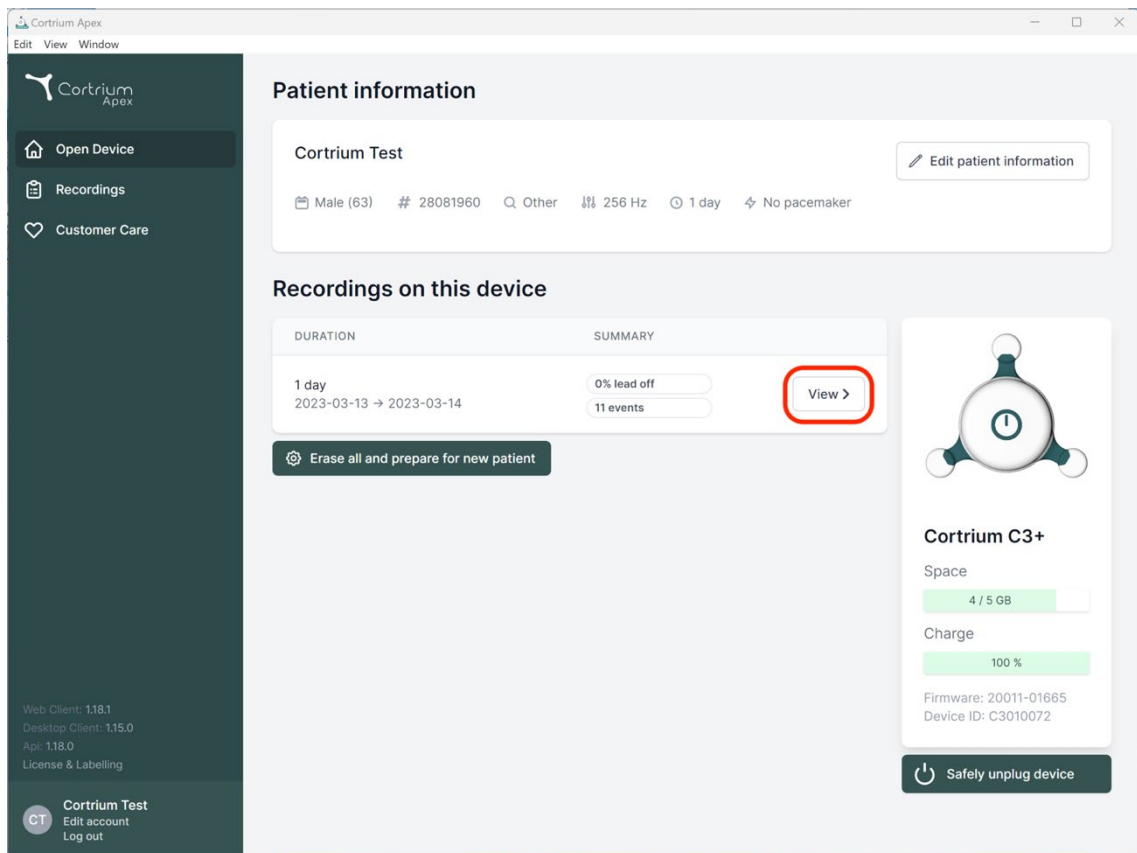
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
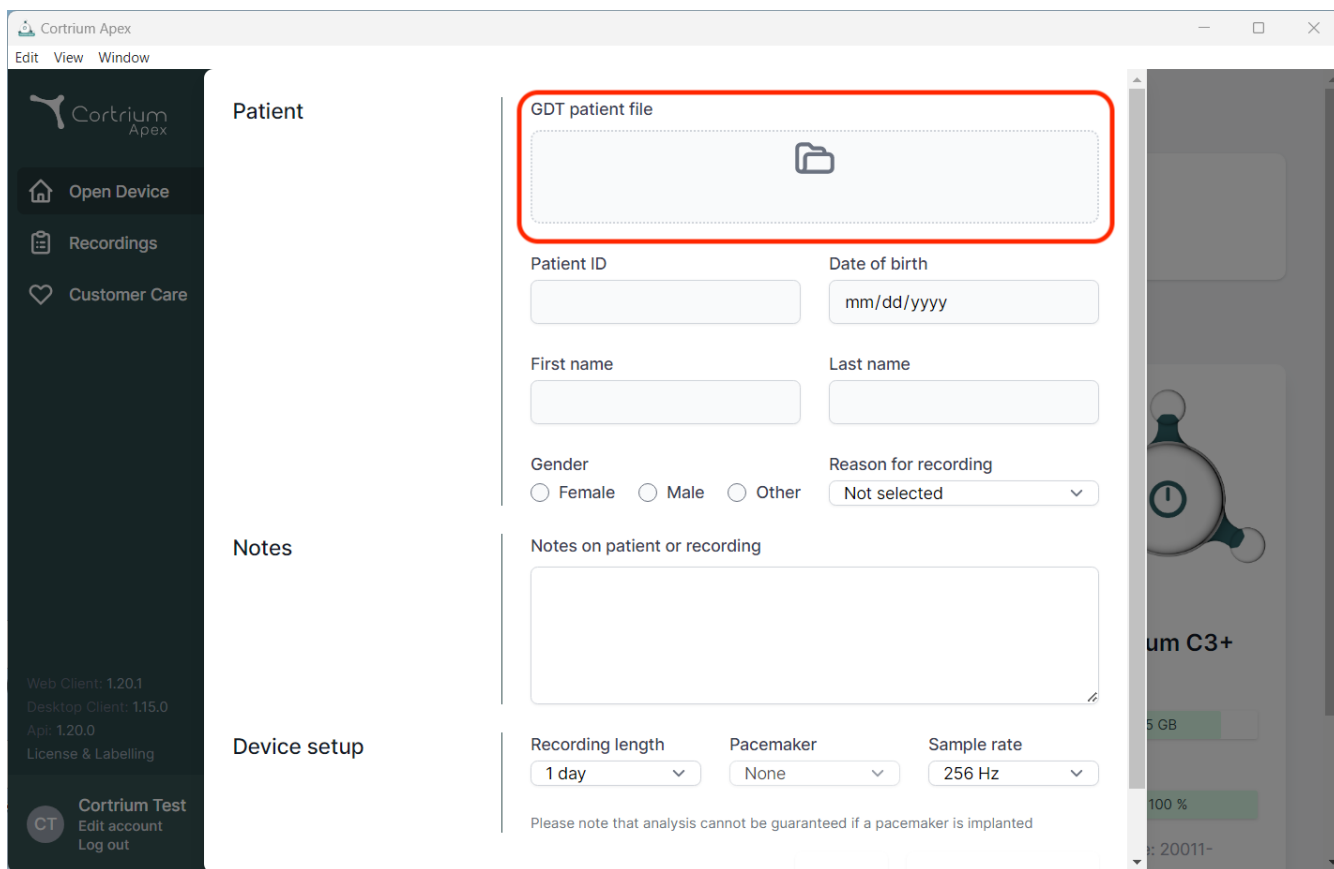
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: Open Device, Recordings, and Customer Care. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, App: 1.20.0) and a user profile section for Cortrium Test. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a red-bordered box labeled 'GDT patient file' with a folder icon, indicating where to upload the patient's GDT file. Below this are input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'.

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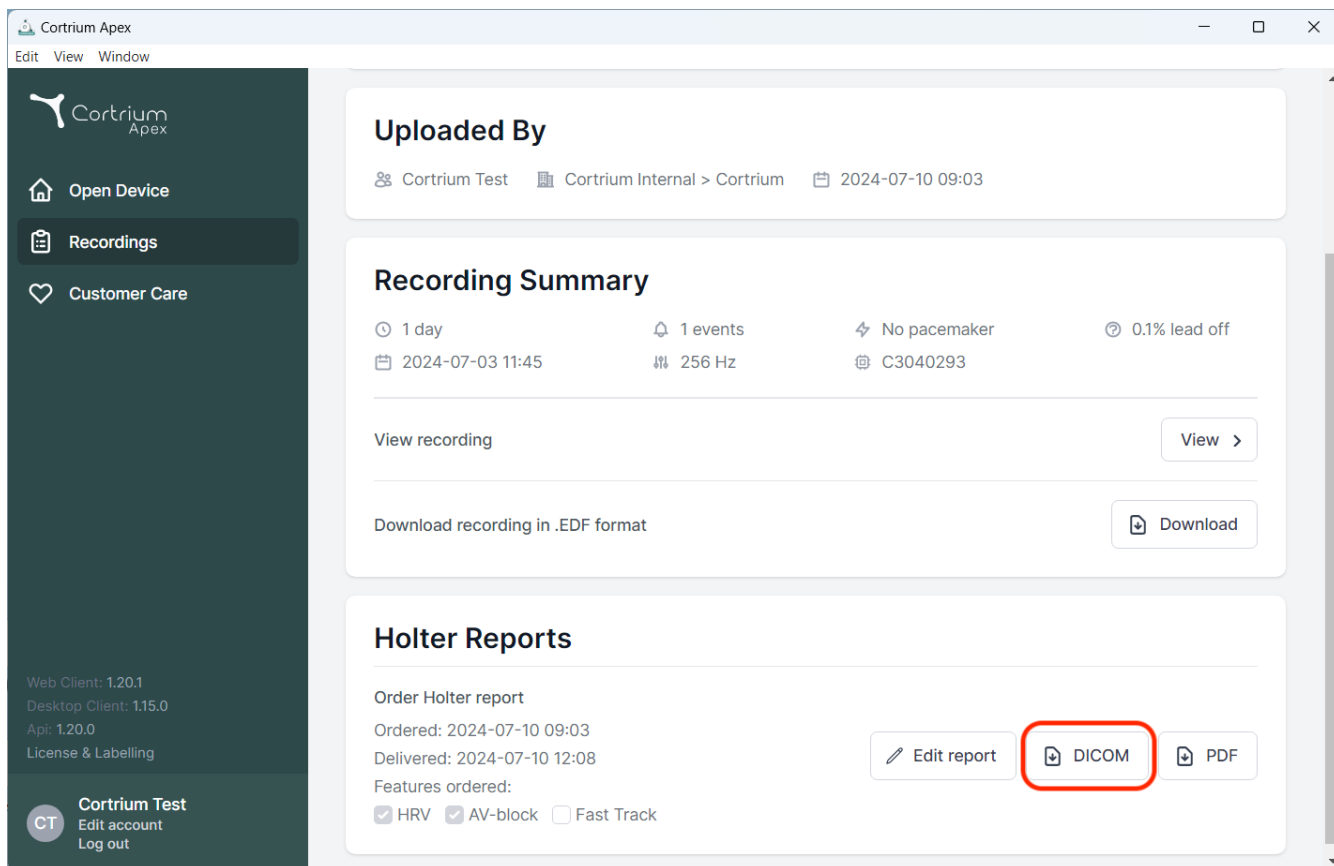
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (for .EDF format).
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

At the bottom left of the sidebar, version information is listed: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. A user profile for 'Cortrium Test' is also visible.

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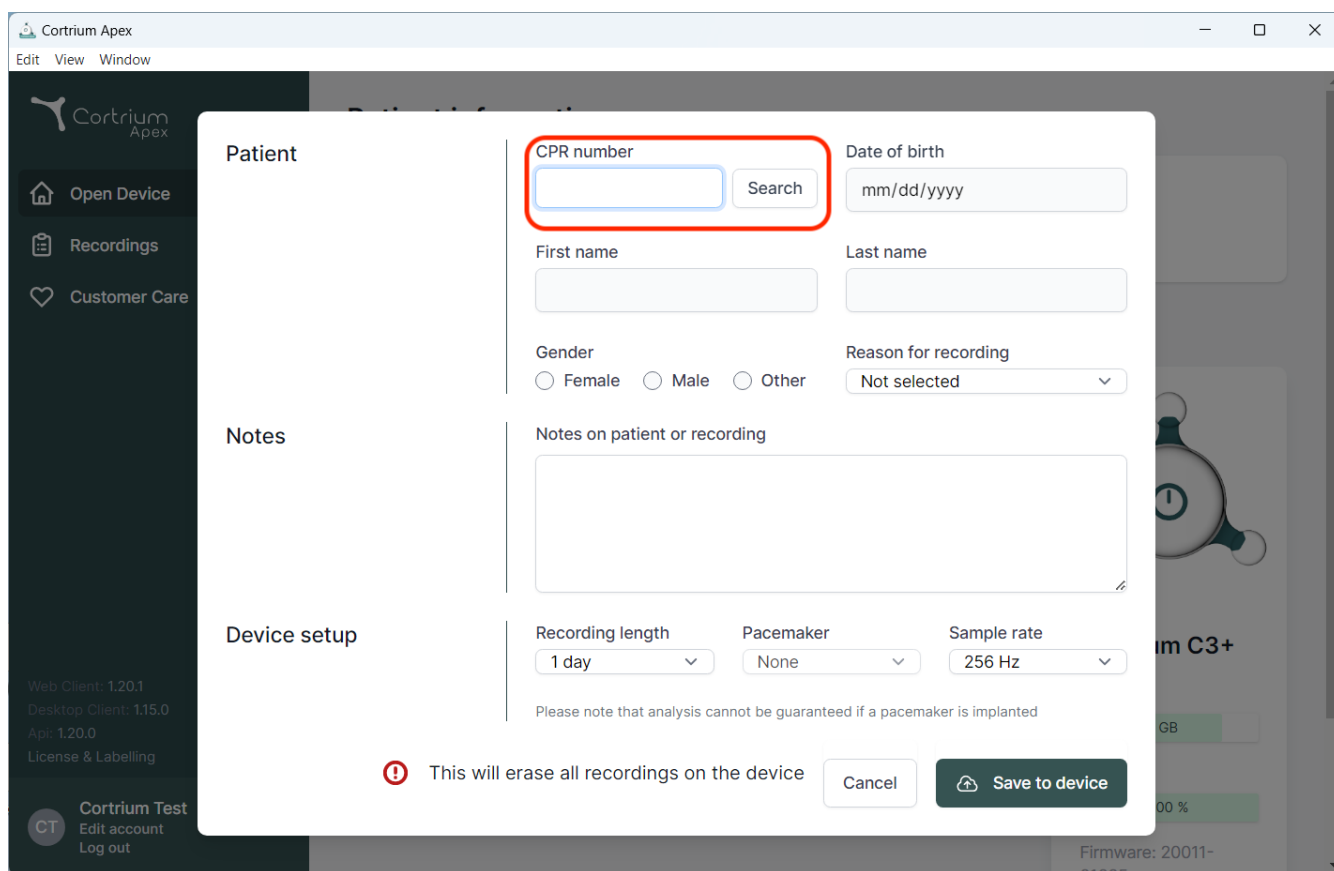
The screenshot displays the Cortrium Apex application window. A modal dialog titled "Patient" is open, allowing for patient search and data entry. The modal is divided into several sections:

- Patient:** Contains input fields for "CPR number" (highlighted with a red box), "Date of birth" (format: mm/dd/yyyy), "First name", and "Last name". There is a "Search" button next to the CPR number field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Includes dropdowns for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup, a warning message states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom of the modal, there is a red warning icon and the text "This will erase all recordings on the device", along with "Cancel" and "Save to device" buttons.

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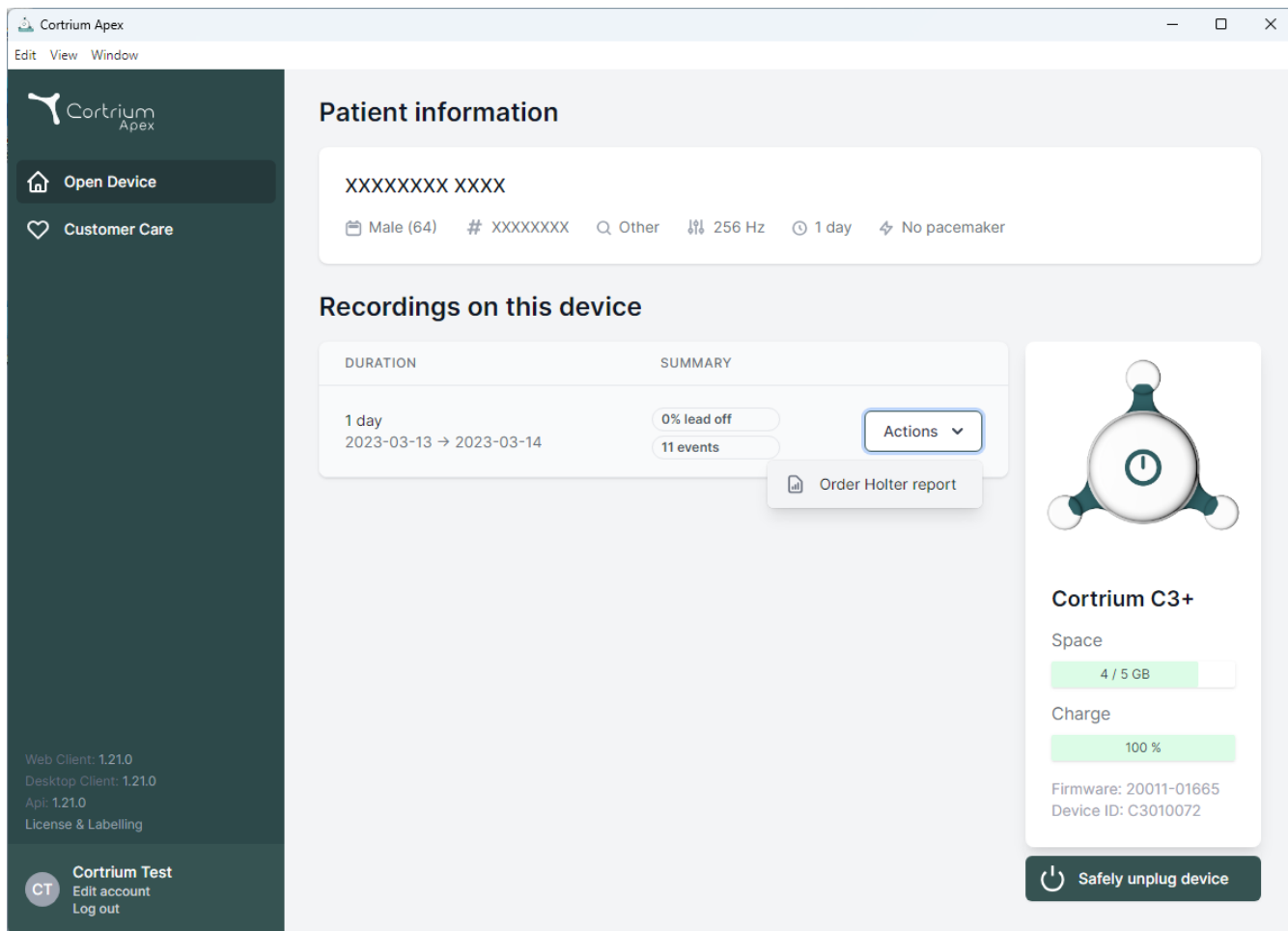
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for a '1 day' recording from '2023-03-13 → 2023-03-14'. The summary for this recording shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side of the interface, there is a section for the 'Cortrium C3+' device, showing its status: 'Space' (4 / 5 GB), 'Charge' (100%), 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of this section.

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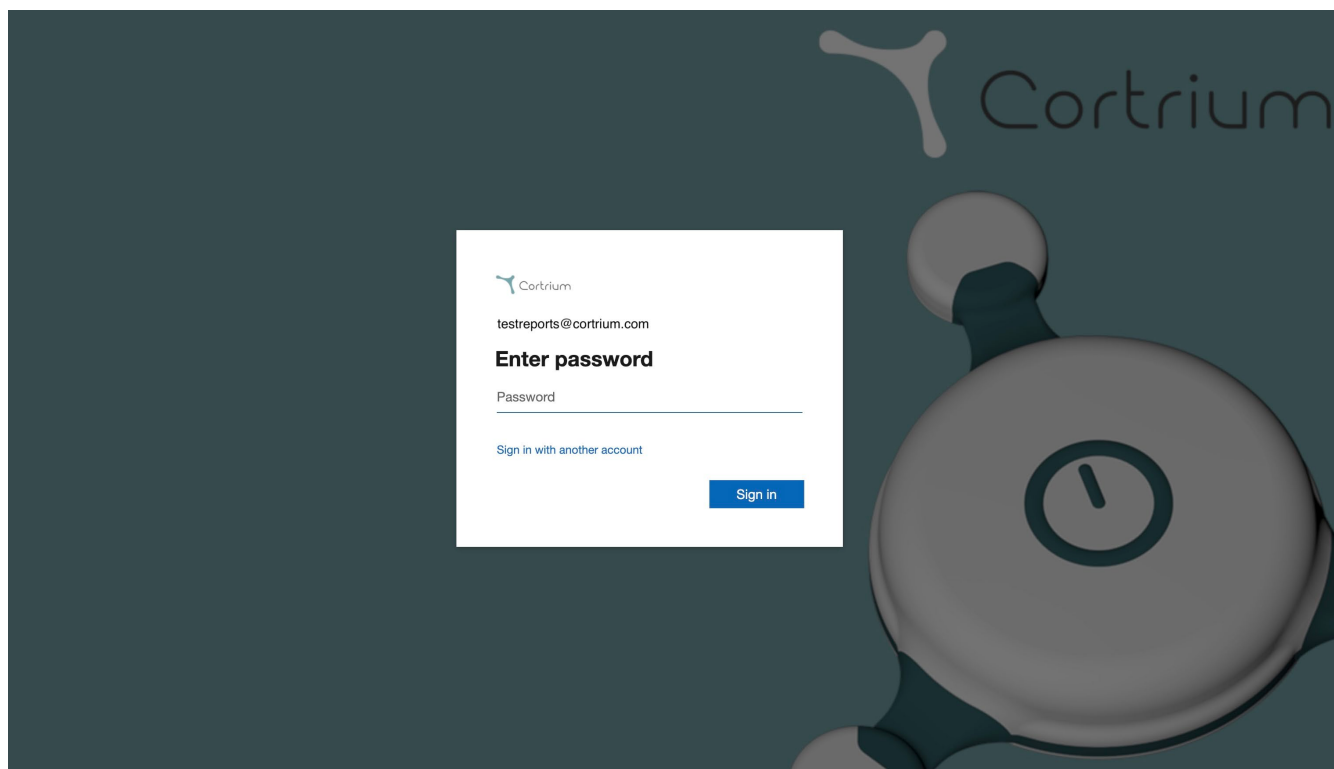
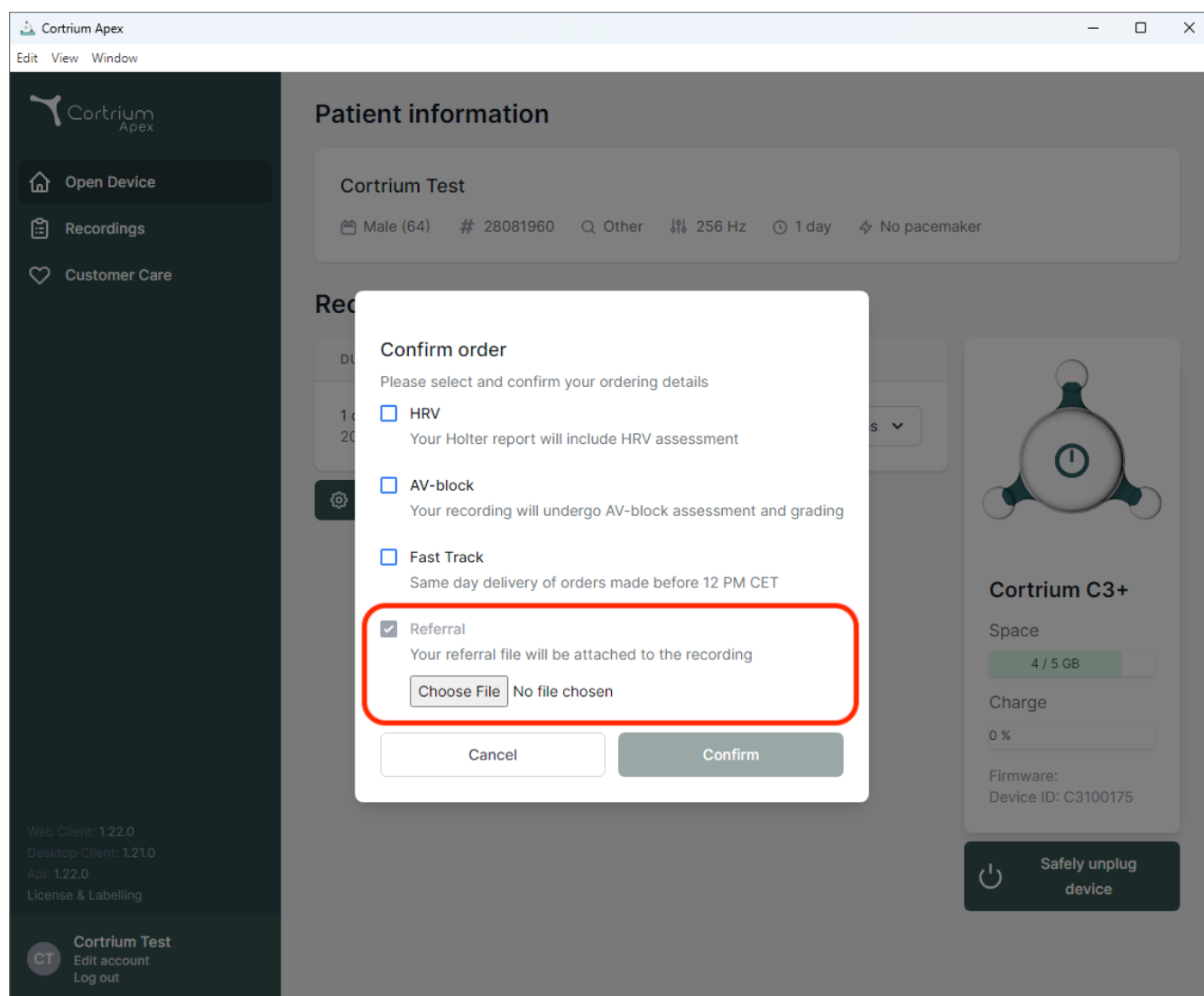
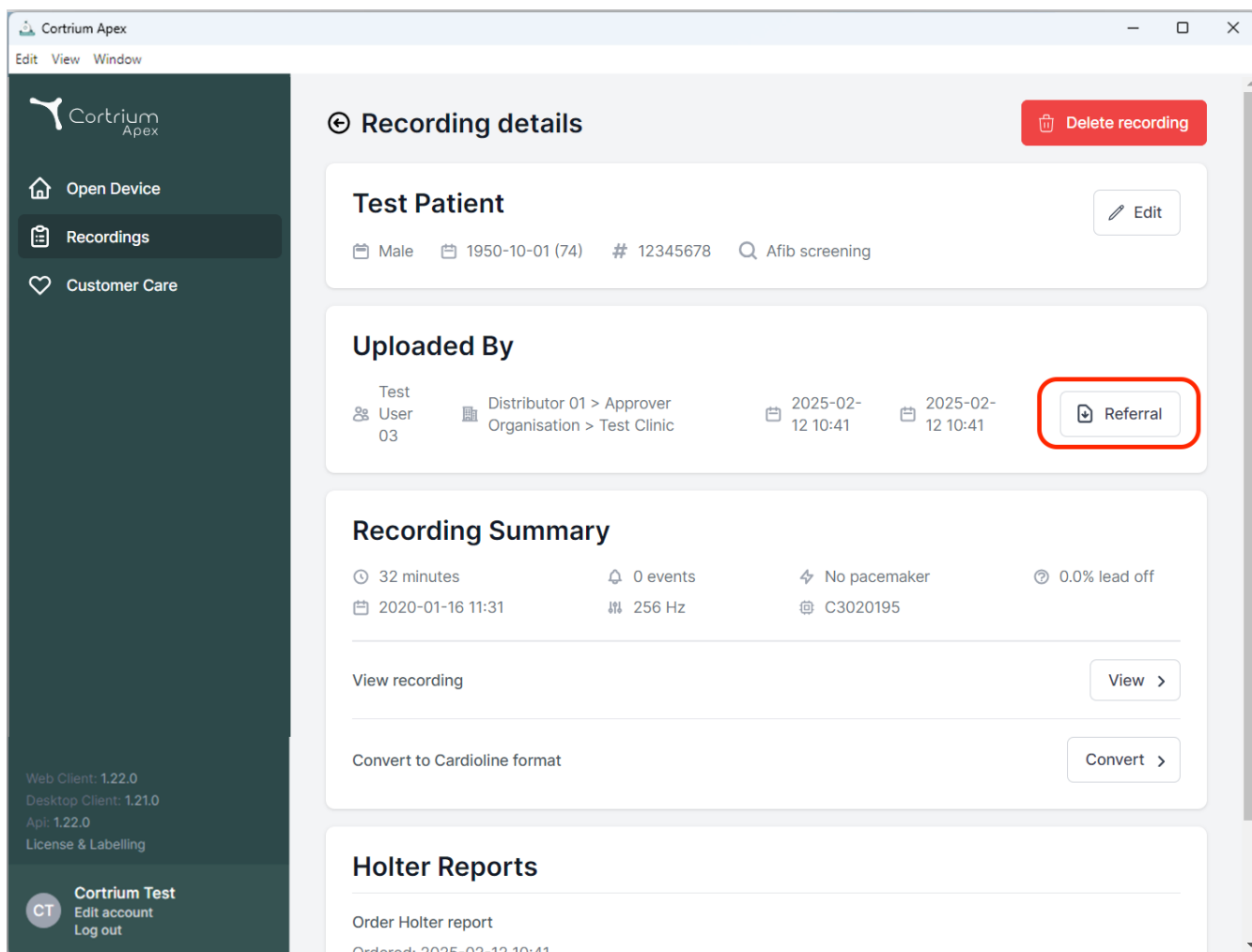
The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right. Below this, the 'Test Patient' section shows details for a male patient born on 1950-10-01 (74 years old), with ID # 12345678 and a note 'Afib screening'. An 'Edit' button is available. The 'Uploaded By' section lists 'Test User 03' as the uploader, associated with 'Distributor 01 > Approver' and 'Organisation > Test Clinic'. It shows two upload dates: 2025-02-12 10:41. A 'Referral' button, highlighted with a red rectangle, is located in this section. The 'Recording Summary' section provides key metrics: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also includes a date '2020-01-16 11:31', a frequency of 256 Hz, and a device ID 'C3020195'. Below the summary are buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section at the bottom includes an 'Order Holter report' button and a partially visible 'Ordered: 2025-02-12 10:41' timestamp. The bottom of the sidebar shows the user 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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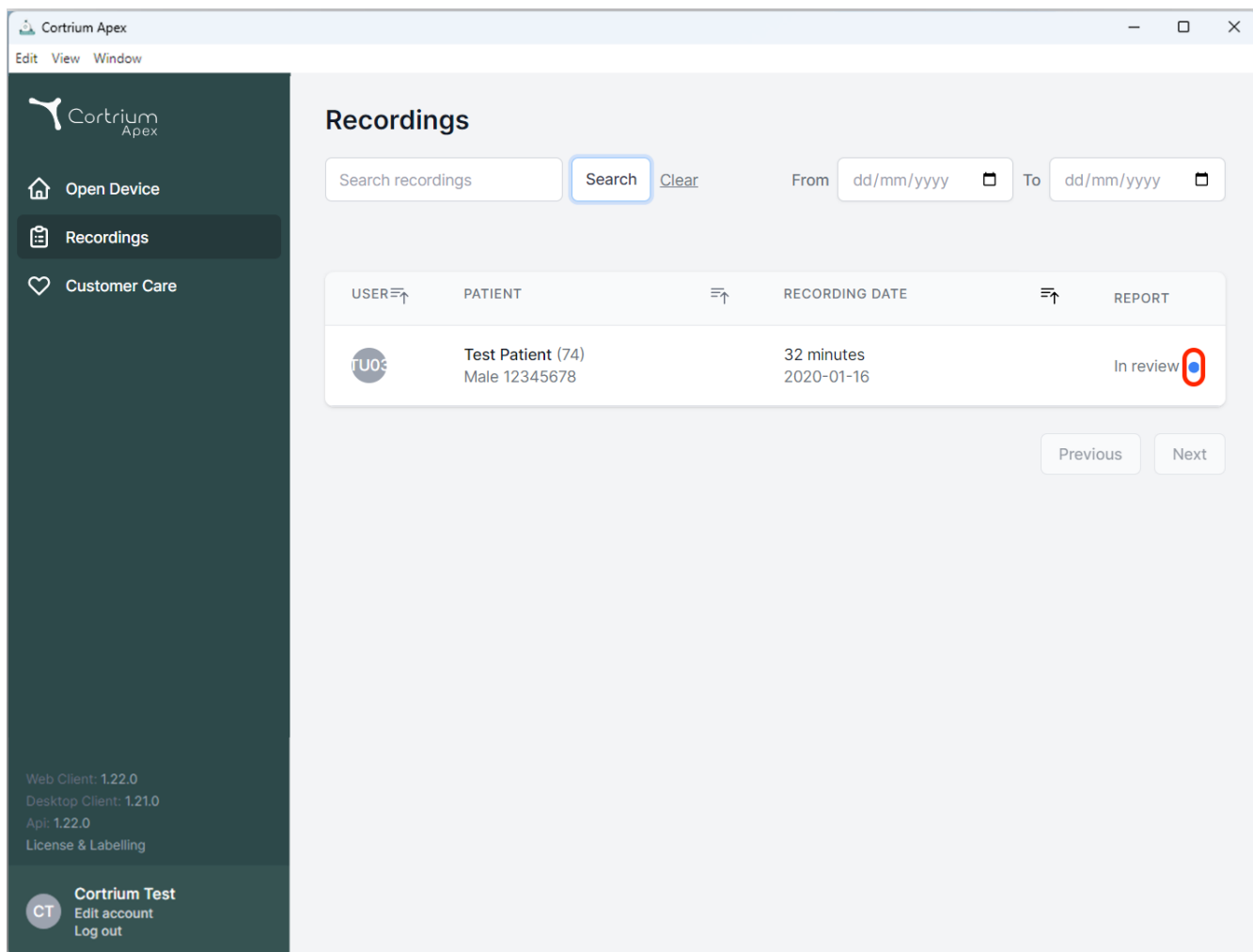The screenshot shows the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area displays a table of recordings. The table has columns for 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. A single recording is listed for 'Test Patient (74)' with a recording date of '2020-01-16' and a duration of '32 minutes'. The 'REPORT' column shows 'In review' with a blue dot icon circled in red. The interface also includes search filters, a 'Search' button, and 'Previous' and 'Next' navigation buttons.

Recordings

Search recordings From To


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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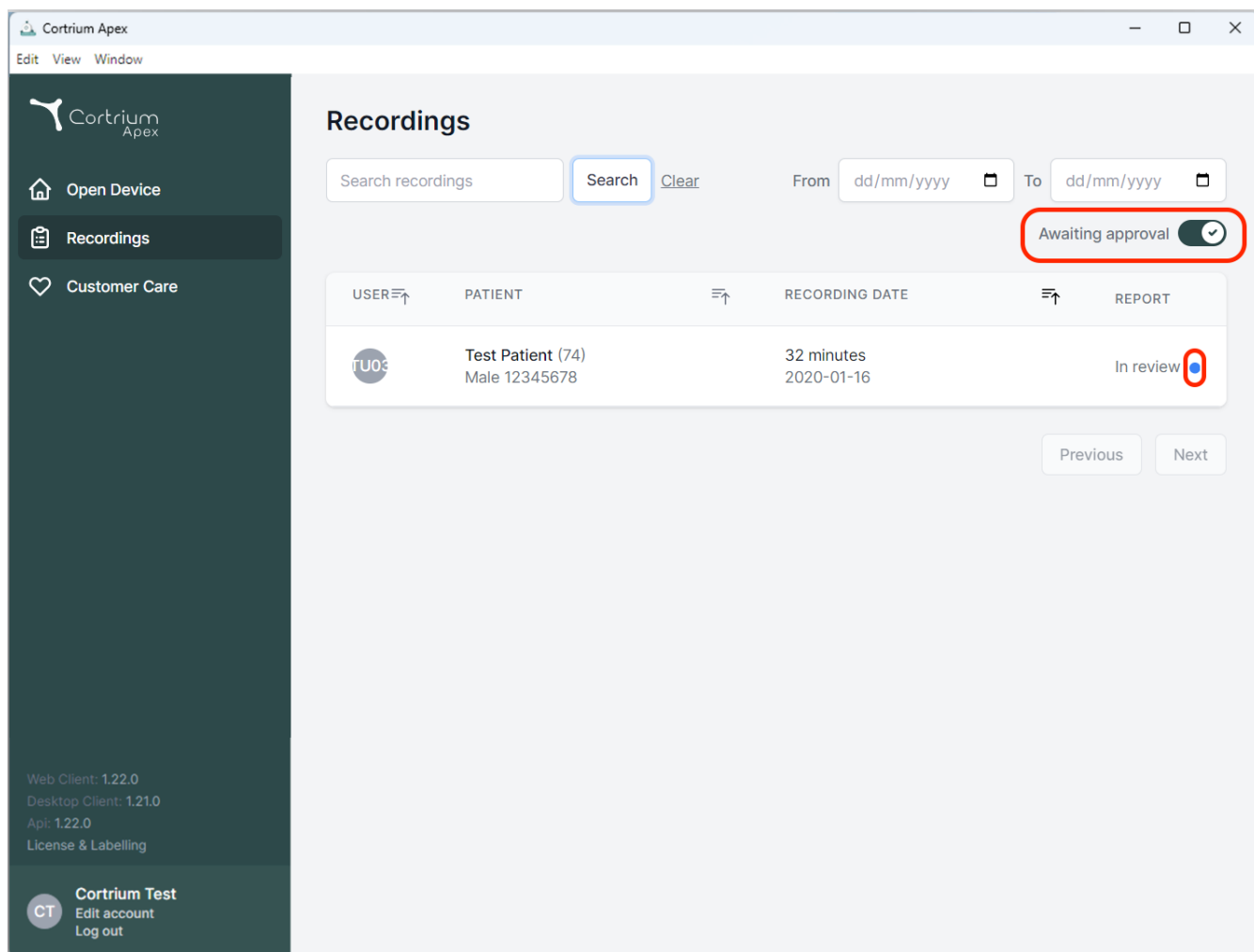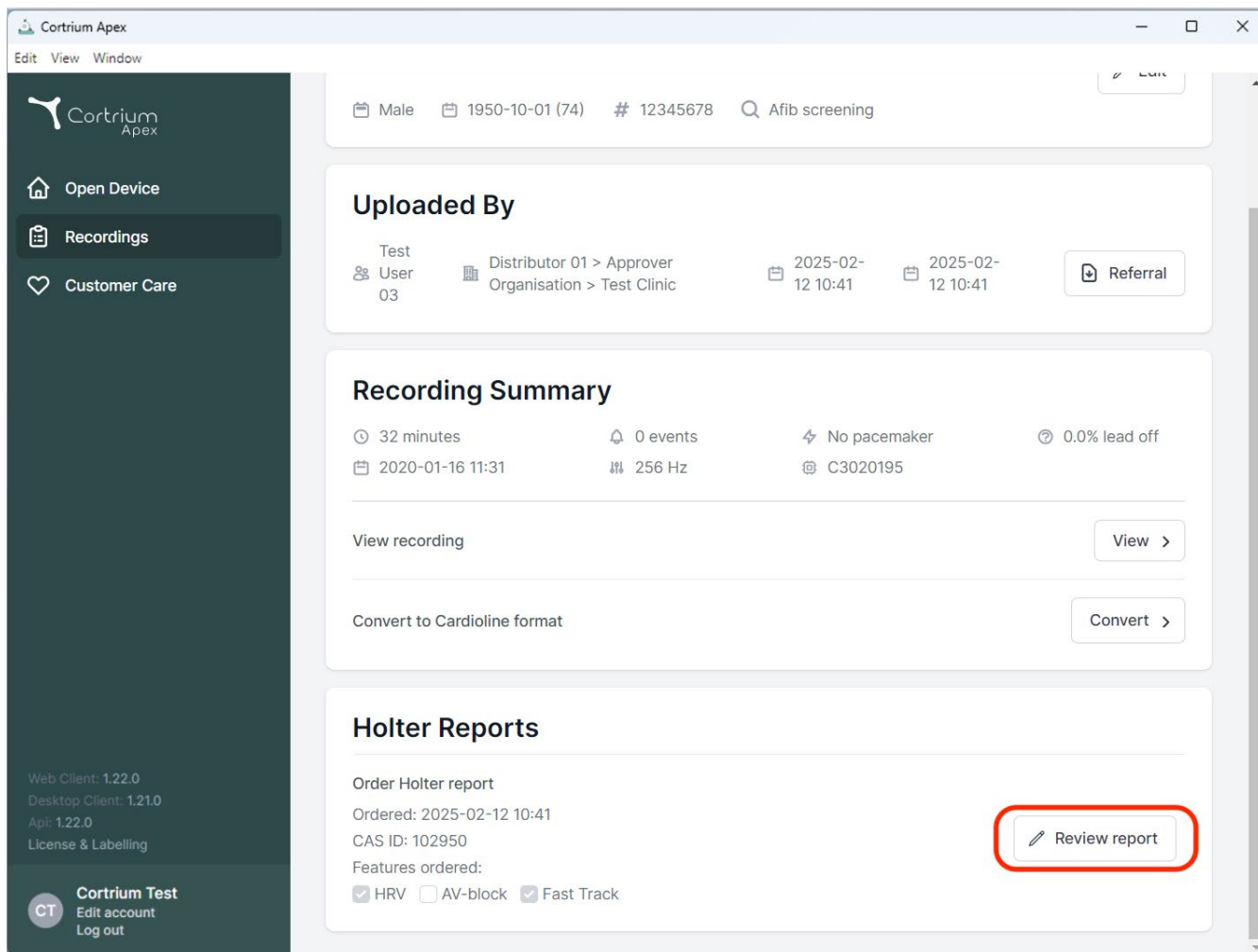
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41 and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

Uploaded By

Test User	Distributor	Approver	Organisation	Test Clinic	Upload Date	Action
03	Distributor 01	Approver	Organisation	Test Clinic	2025-02-12 10:41	Referral

Recording Summary

Duration	Events	Pacemaker	Lead off	Date	Frequency	ID
32 minutes	0 events	No pacemaker	0.0% lead off	2020-01-16 11:31	256 Hz	C3020195

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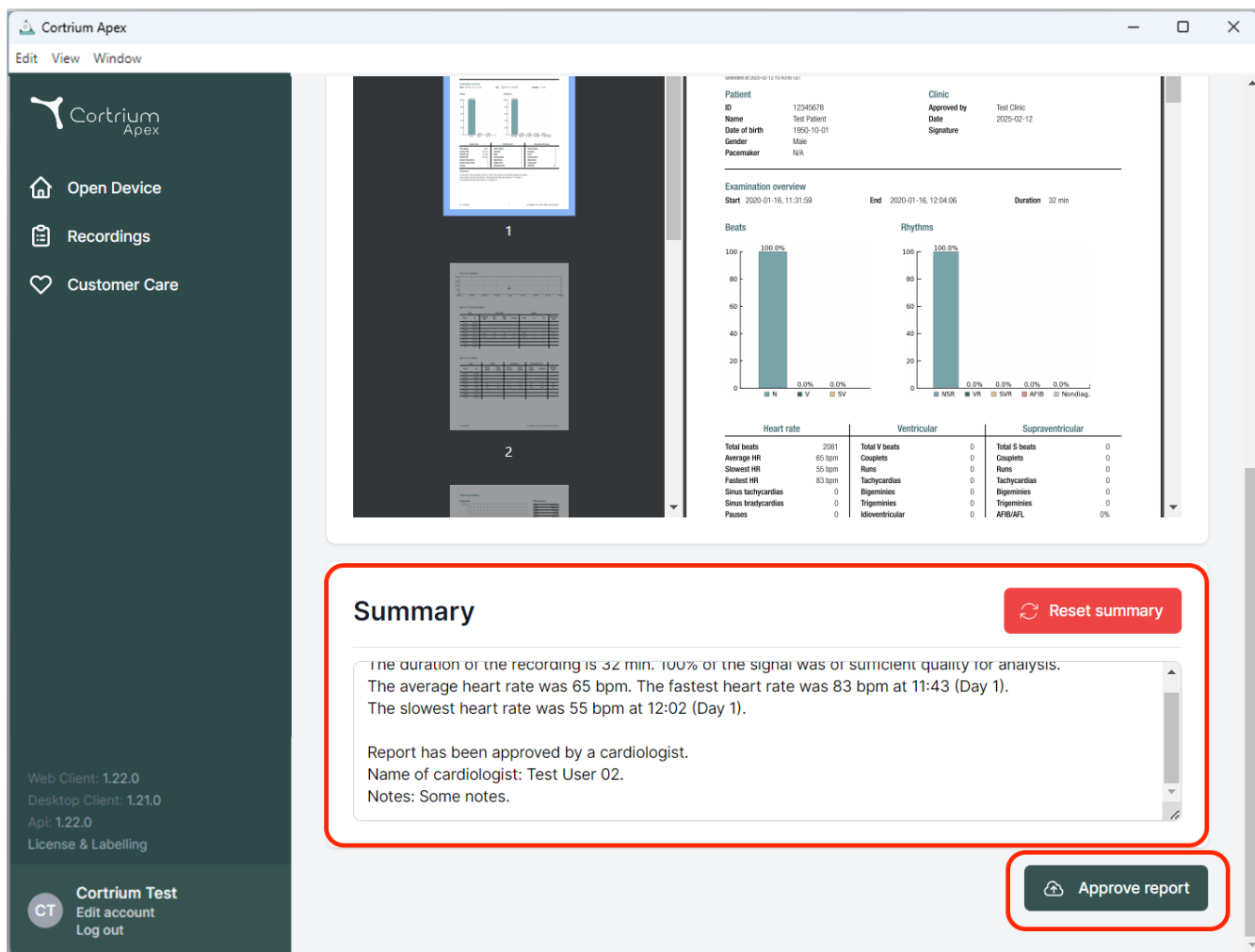
Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

Beats
100.0%
0.0% 0.0% 0.0%

Rhythms
100.0%
0.0% 0.0% 0.0% 0.0% 0.0% 0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AF/IB/ATL	0%

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dania
E-mail: info@cortrium.com • Strona internetowa: www.cortrium.com

Cortrium Apex UDI(01)05745000379033