# Cortrium Apex 1.22

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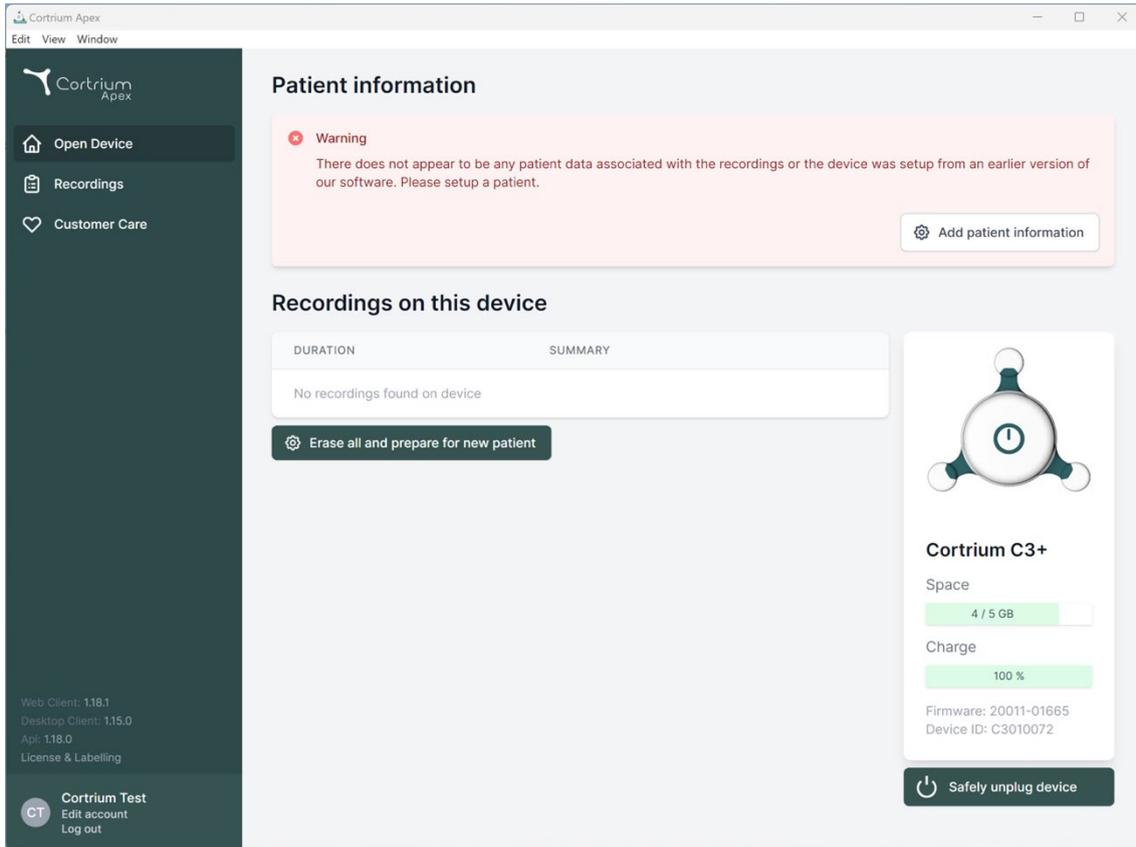
Cortrium Test  
Edit account  
Log out

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.

Add patient information

### Recordings on this device

DURATION	SUMMARY
No recordings found on device	

Erase all and prepare for new patient

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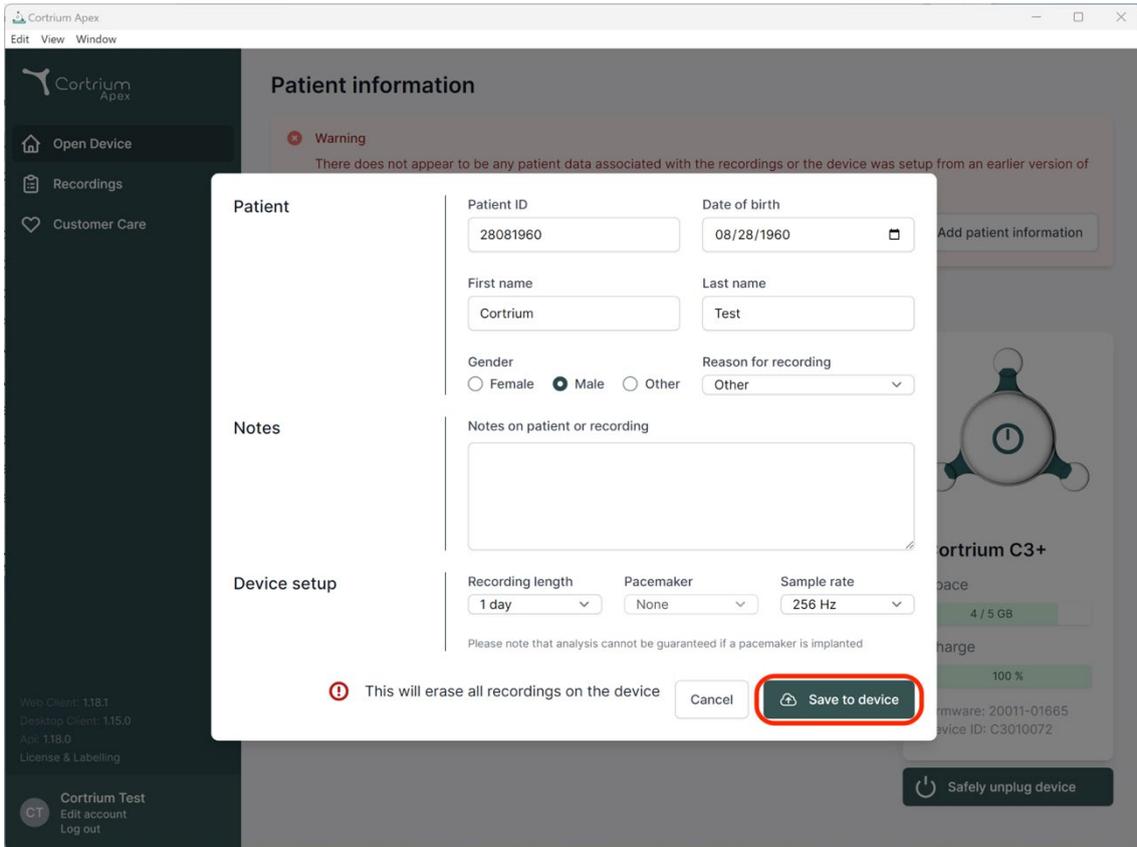
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient information" is open, displaying a form for entering patient details. The form is divided into three sections: "Patient", "Notes", and "Device setup".

**Patient information:**

- Patient ID:** 28081960
- Date of birth:** 08/28/1960
- First name:** Cortrium
- Last name:** Test
- Gender:**  Female  Male  Other
- Reason for recording:** Other

**Notes:**

Notes on patient or recording

**Device setup:**

- Recording length:** 1 day
- Pacemaker:** None
- Sample rate:** 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**Warning:** There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Buttons:** Cancel, Save to device (highlighted with a red circle)

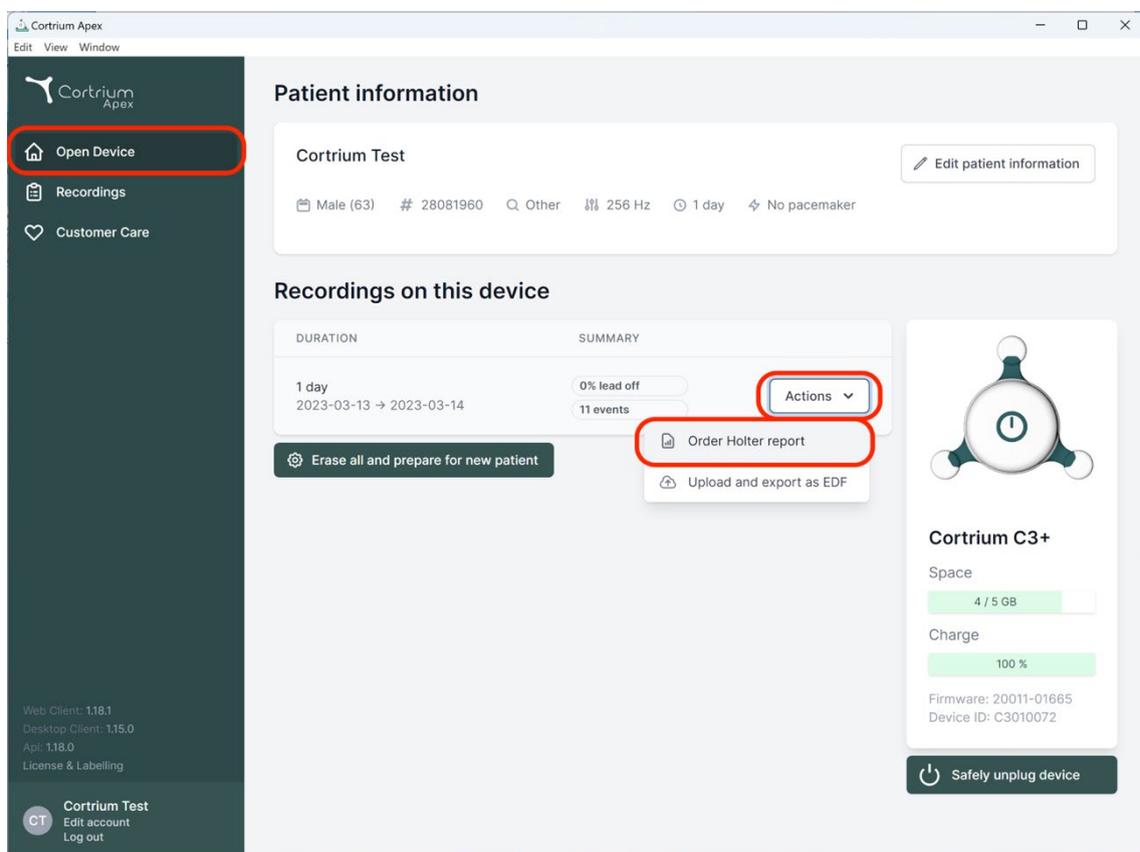
**Footer:** Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labeling, Cortrium Test, Edit account, Log out

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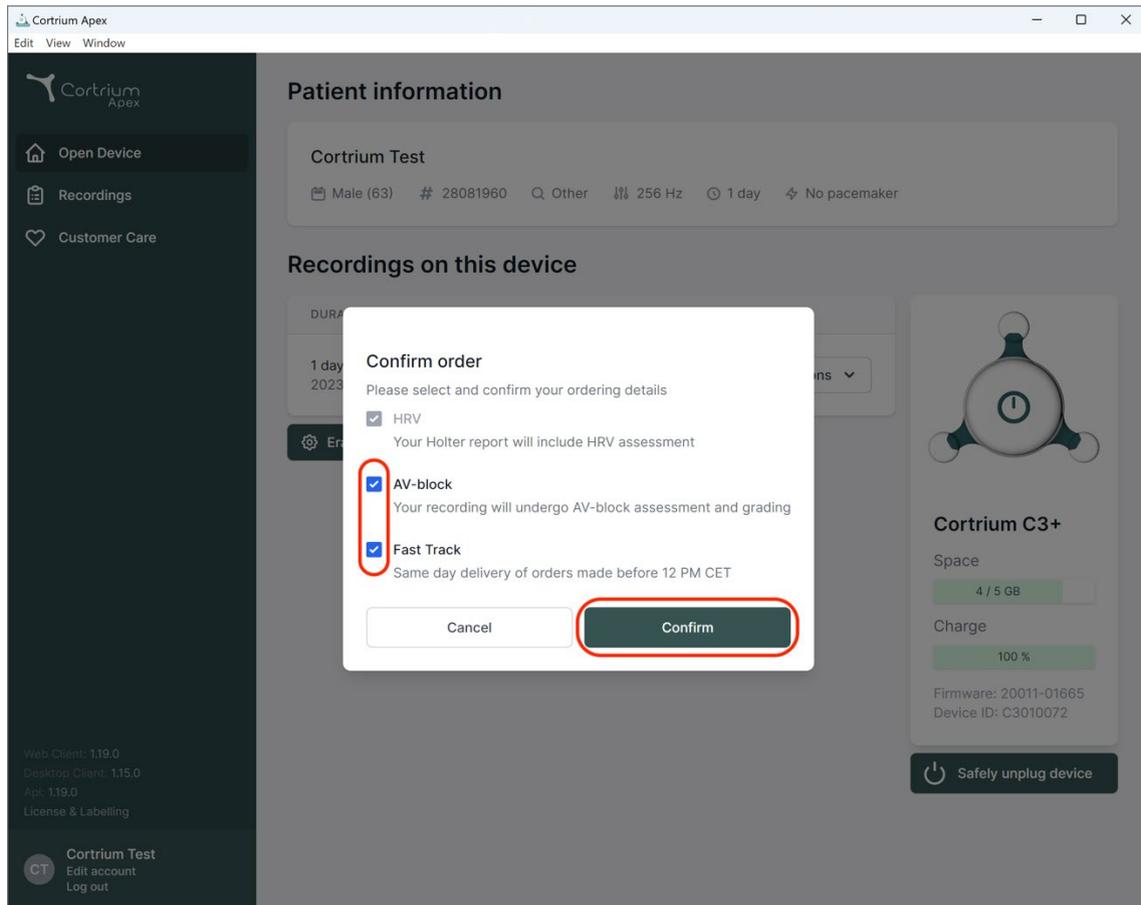
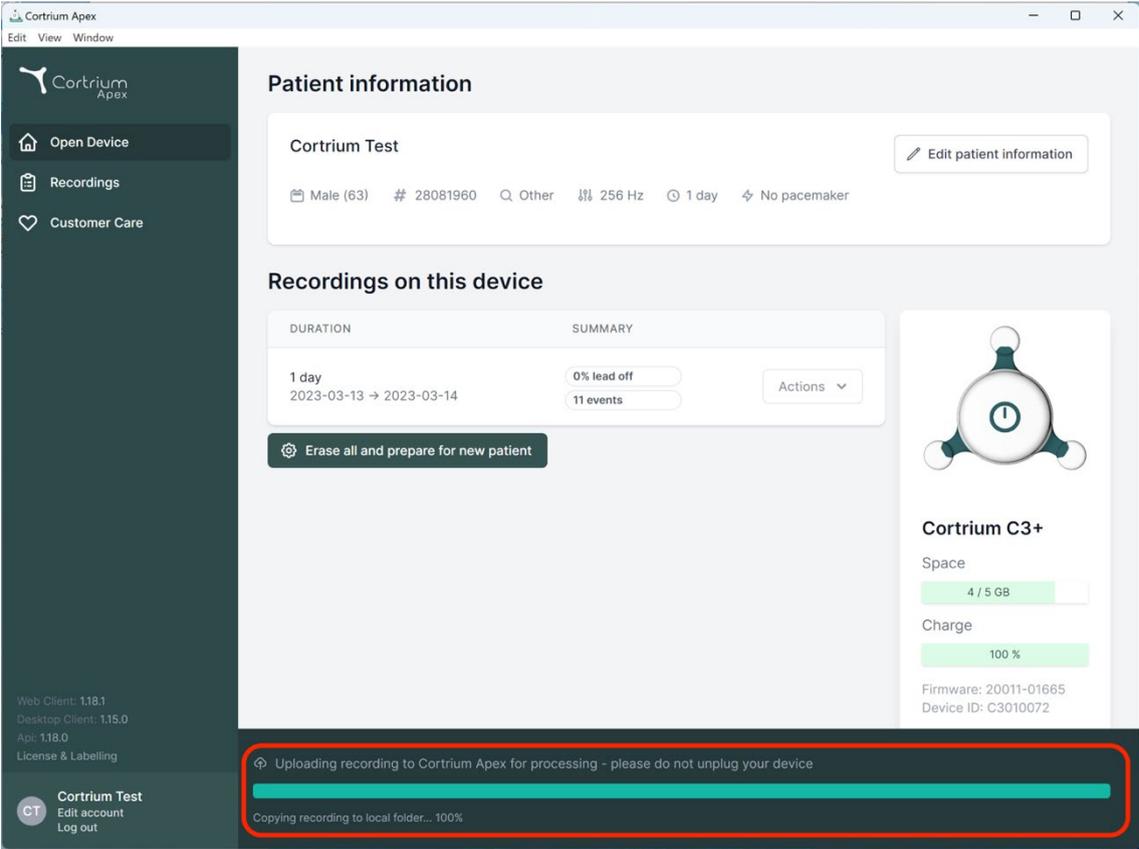
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and test parameters (Other, 256 Hz, 1 day, No pacemaker). Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A 'Erase all and prepare for new patient' button is visible. To the right, a device status card for 'Cortrium C3+' shows 4/5 GB space, 100% charge, and firmware/device IDs. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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
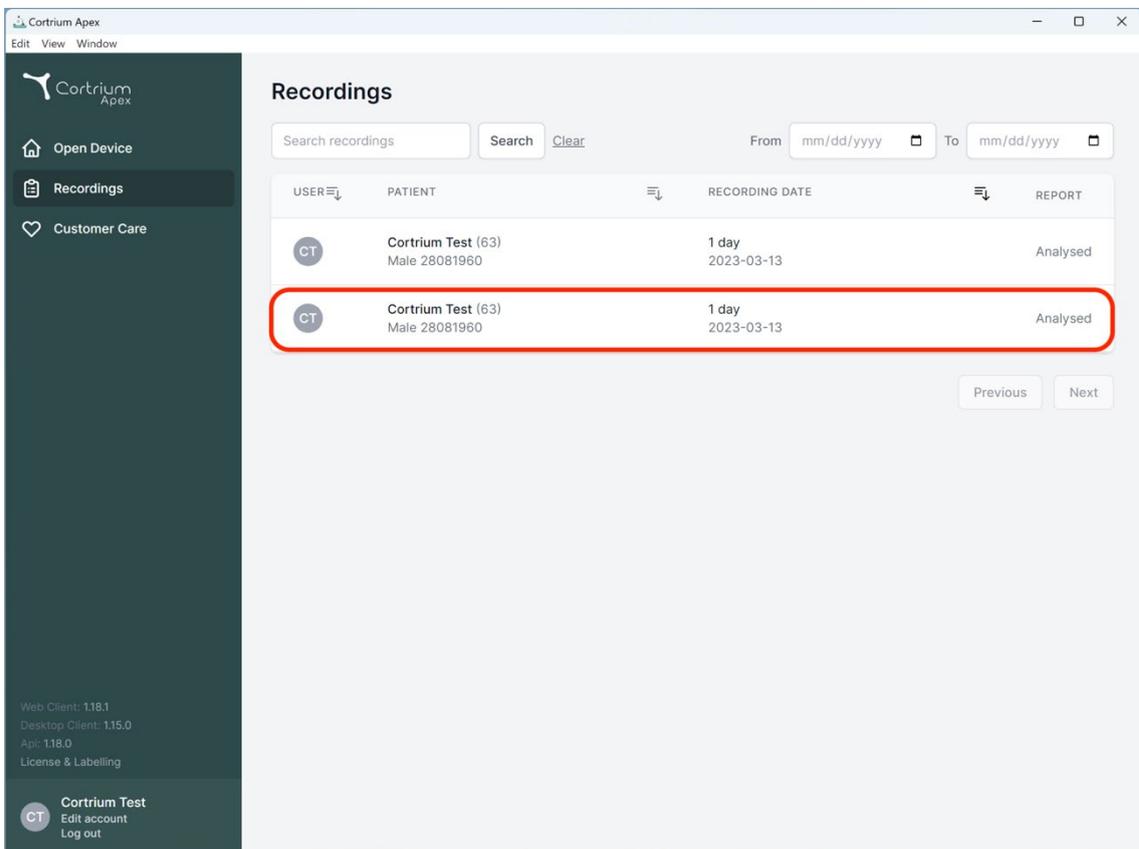
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings (selected), and Customer Care. The main content area is titled "Recordings" and features a search bar with "Search" and "Clear" buttons, and date filters for "From" and "To". Below this is a table with the following data:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

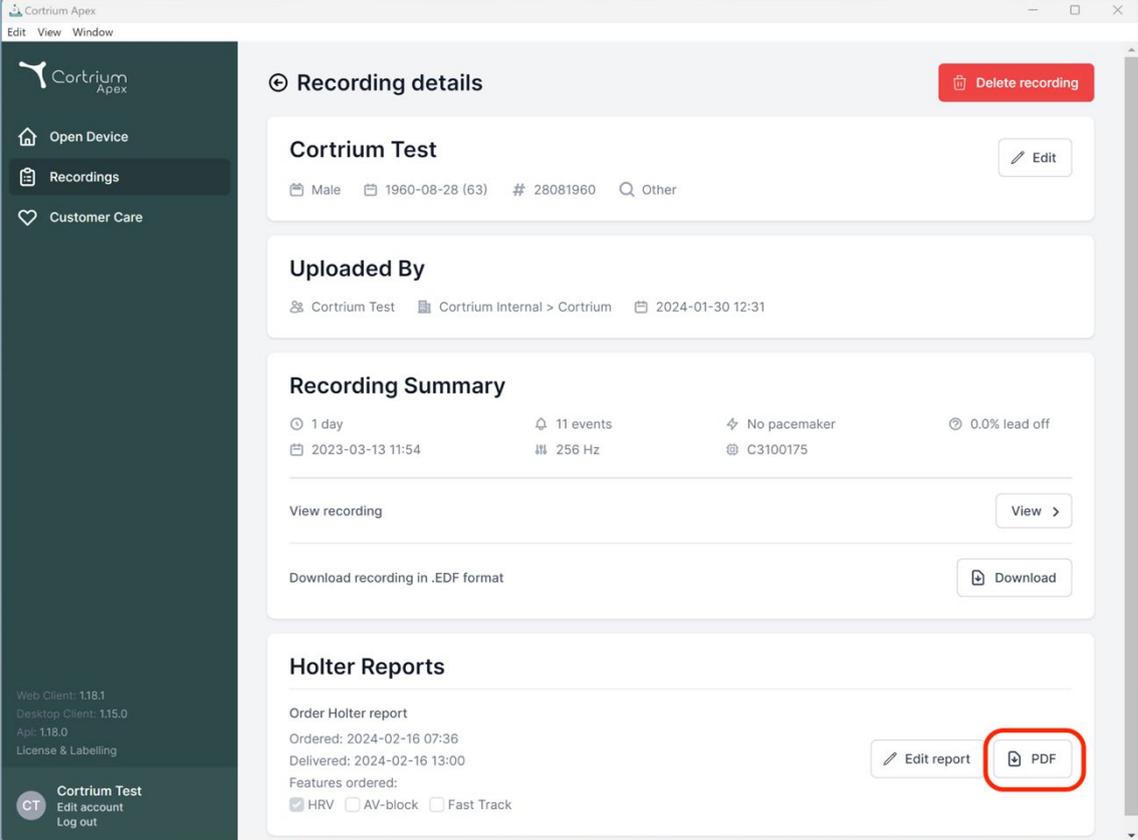The second row of the table is highlighted with a red border. At the bottom right of the table area are "Previous" and "Next" buttons. The footer of the sidebar includes version information (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Recording details' and features a red 'Delete recording' button. Below this, the recording is identified as 'Cortrium Test' with an 'Edit' button. Patient information includes 'Male', '1960-08-28 (63)', '# 28081960', and 'Other'. The 'Uploaded By' section shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with an upload date of '2024-01-30 12:31'. The 'Recording Summary' section provides key metrics: '1 day' duration, '2023-03-13 11:54' date, '11 events', '256 Hz' sampling rate, 'No pacemaker', and '0.0% lead off'. It includes 'View recording' and 'Download recording in .EDF format' options. The 'Holter Reports' section shows an 'Order Holter report' with 'Ordered: 2024-02-16 07:36' and 'Delivered: 2024-02-16 13:00'. Features ordered include 'HRV' (checked), 'AV-block', and 'Fast Track'. 'Edit report' and 'PDF' buttons are visible, with the 'PDF' button highlighted by a red circle.

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
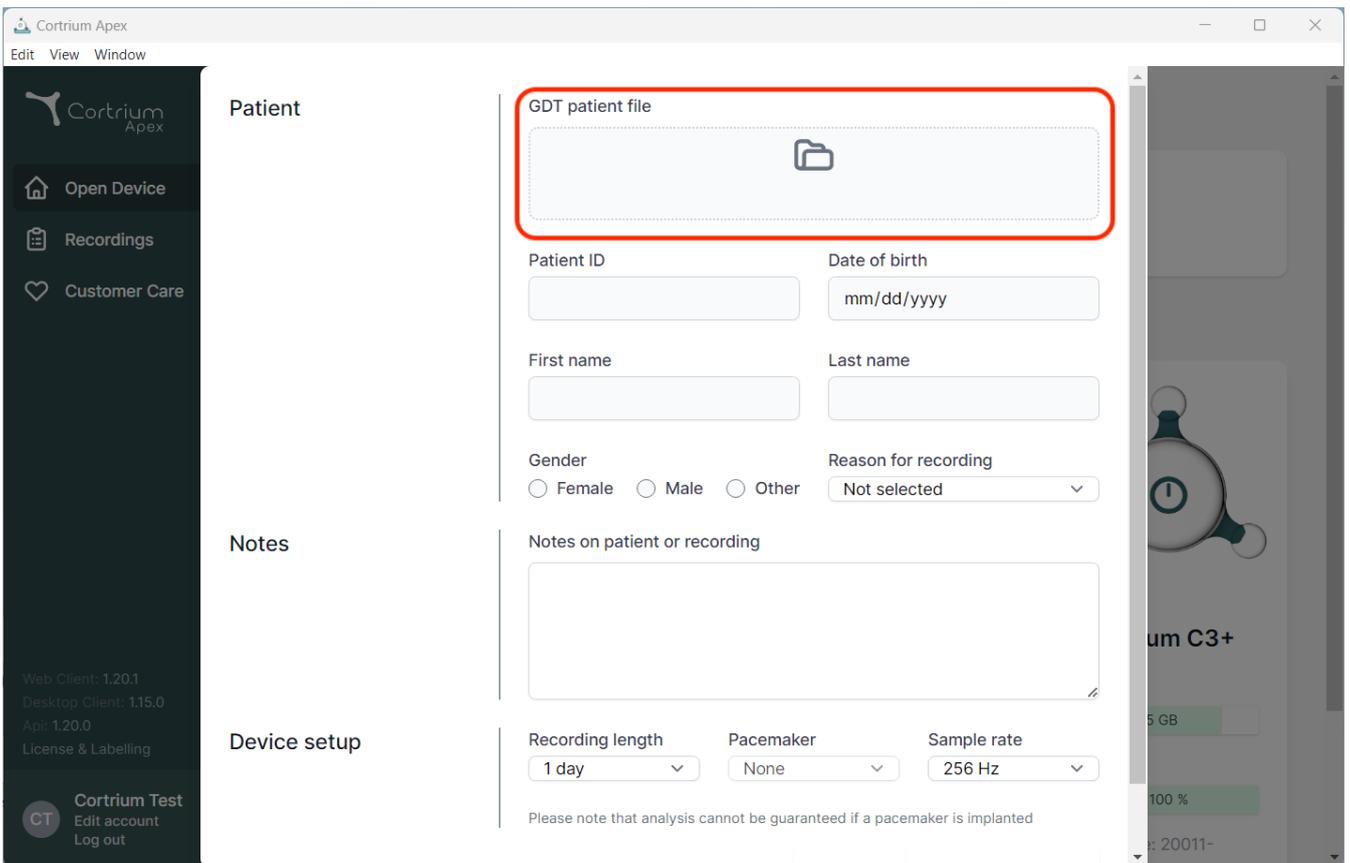
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" field, which is a file upload area. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently set to "Not selected"). There is also a "Notes on patient or recording" text area. At the bottom, the "Device setup" section includes "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner shows version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". The bottom right corner shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

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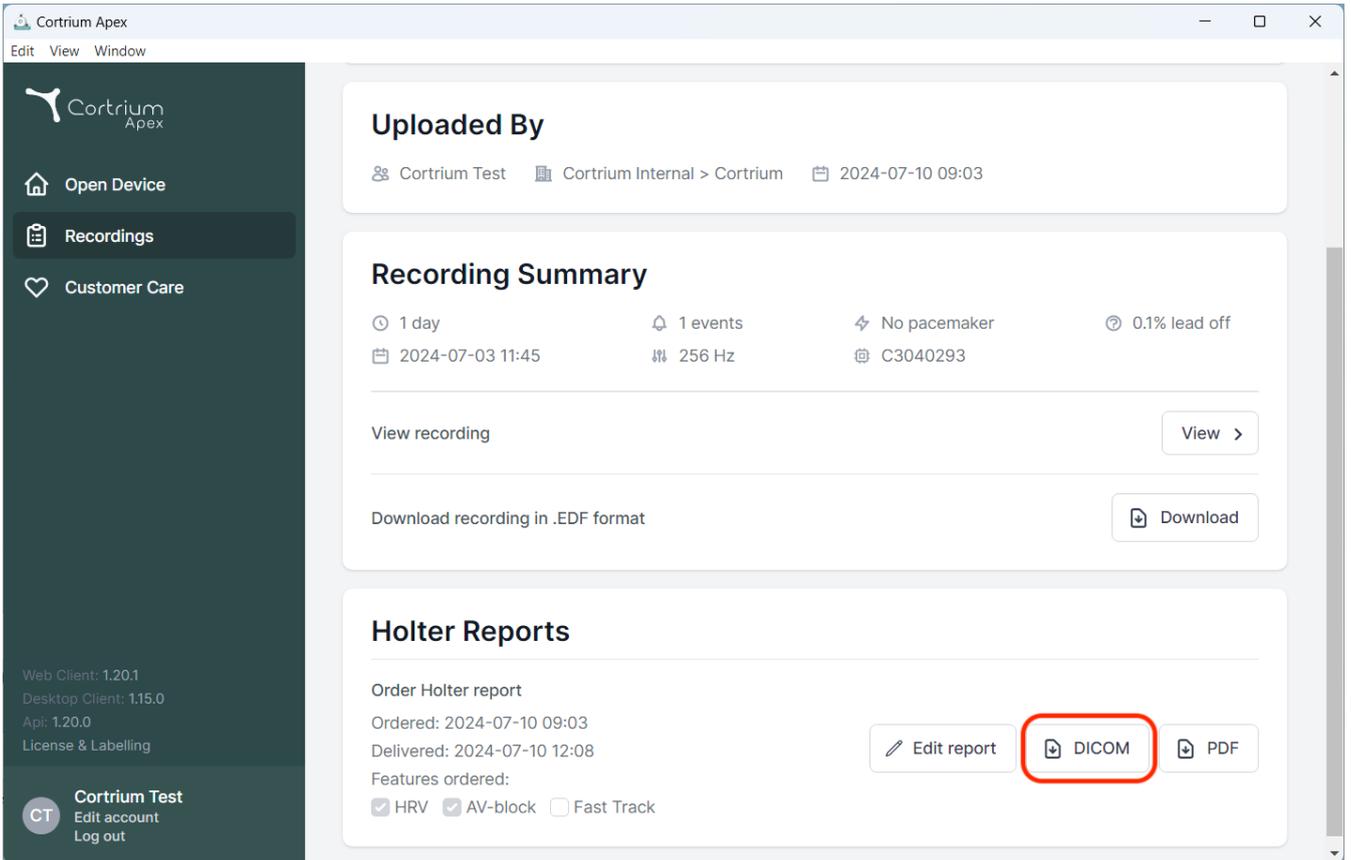
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'. The 'Holter Reports' section shows an 'Order Holter report' with details: 'Ordered: 2024-07-10 09:03', 'Delivered: 2024-07-10 12:08', and 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right of this section, there are three buttons: 'Edit report', 'DICOM', and 'PDF'. The 'DICOM' button is highlighted with a red circle.

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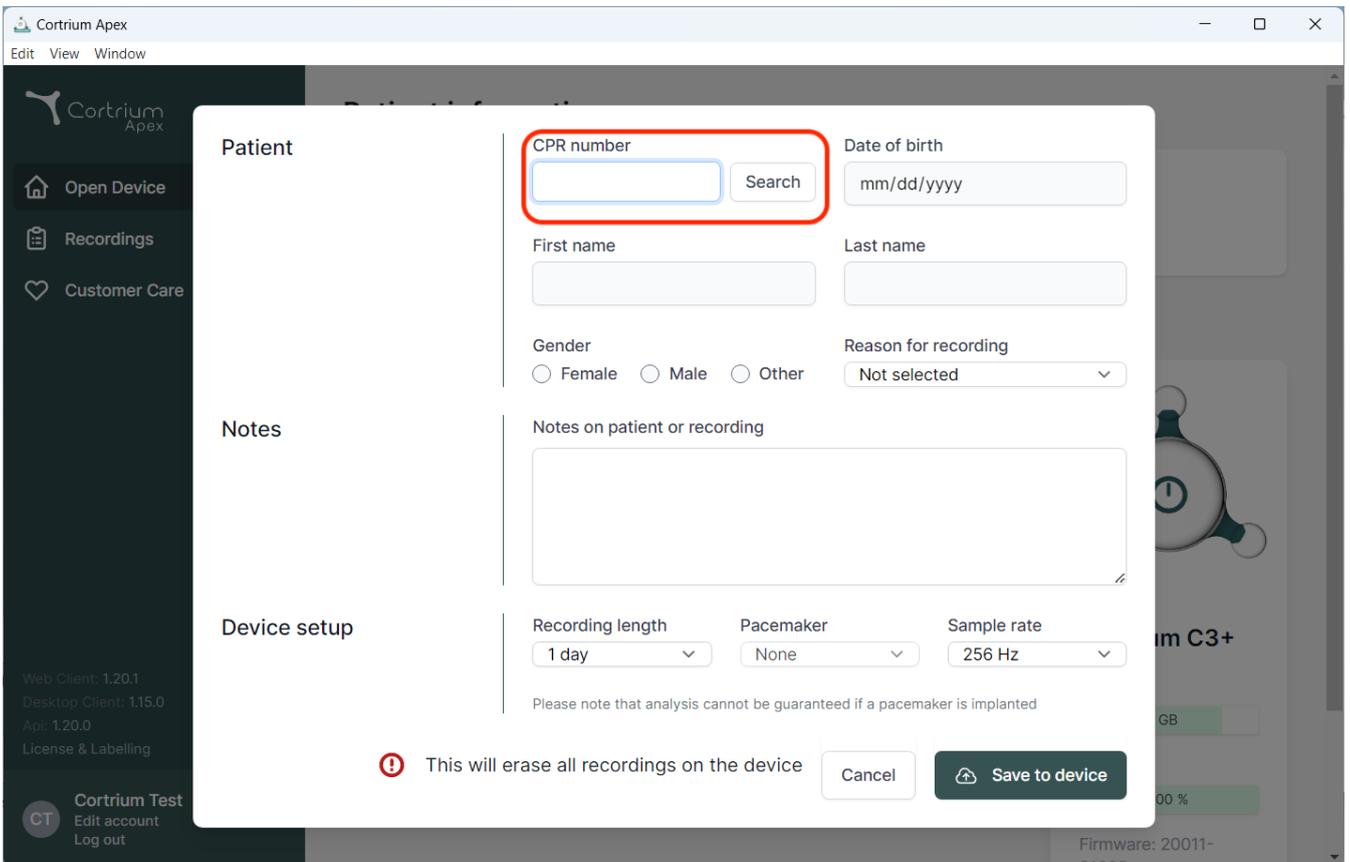
The screenshot displays the 'Cortrium Apex' software interface. A modal window titled 'Patient' is open, containing the following fields and options:

- CPR number:** A text input field with a 'Search' button next to it. This field is highlighted with a red border.
- Date of birth:** A text input field with the placeholder 'mm/dd/yyyy'.
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for 'Female', 'Male', and 'Other'.
- Reason for recording:** A dropdown menu currently showing 'Not selected'.
- Notes:** A large text area labeled 'Notes on patient or recording'.
- Device setup:** Three dropdown menus for 'Recording length' (set to '1 day'), 'Pacemaker' (set to 'None'), and 'Sample rate' (set to '256 Hz').

Below the device setup fields, a warning message is displayed: **!** This will erase all recordings on the device. At the bottom right of the modal, there are 'Cancel' and 'Save to device' buttons.

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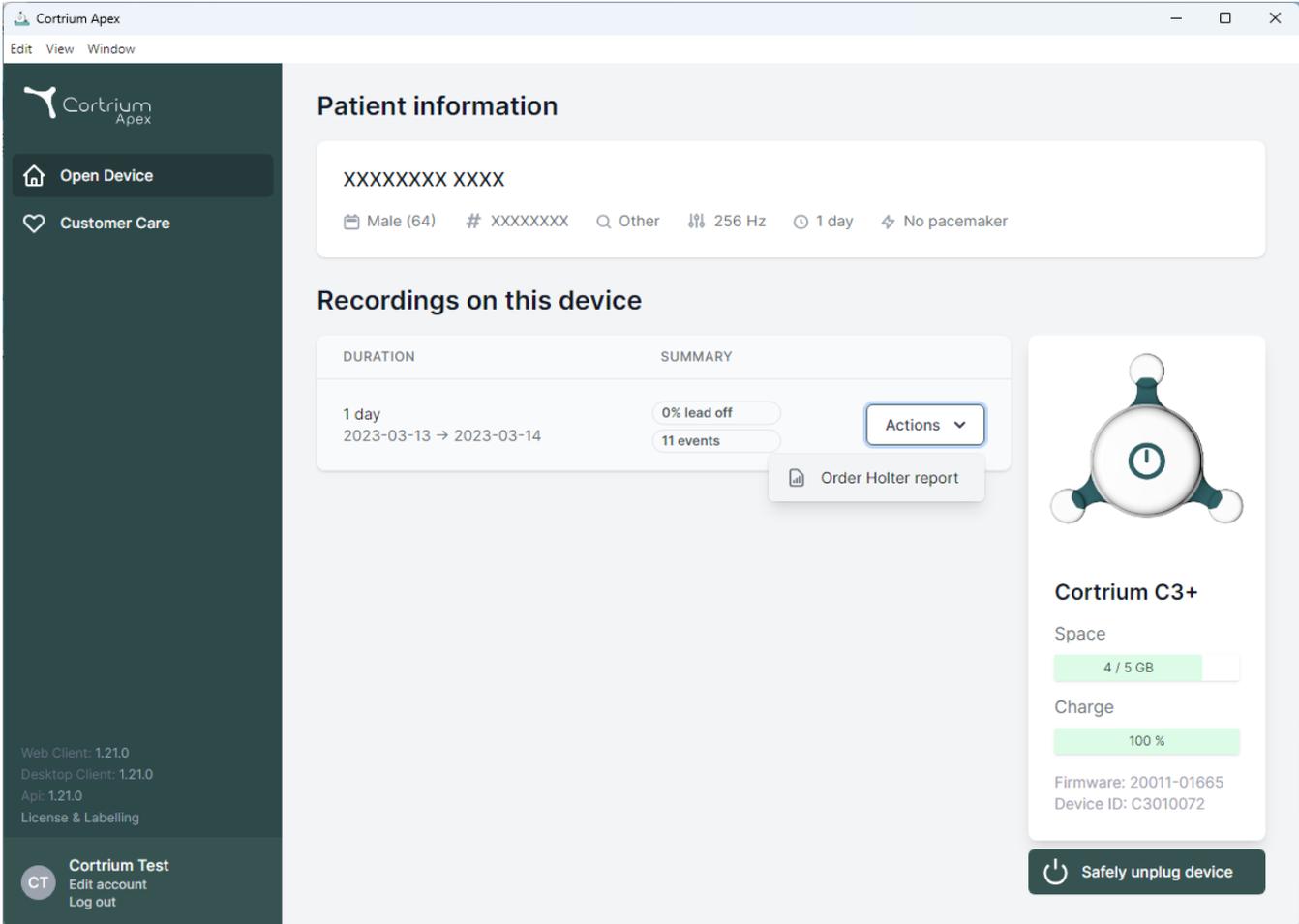
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window'. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' and 'Customer Care'. At the bottom of the sidebar, there are version details for the Web Client, Desktop Client, and API, along with a 'License & Labelling' link. A user profile section for 'Cortrium Test' is visible at the bottom left, with options to 'Edit account' and 'Log out'.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with columns for 'DURATION' and 'SUMMARY'. A recording entry for '1 day' (2023-03-13 to 2023-03-14) shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, showing the option 'Order Holter report'. On the right, a device status card for 'Cortrium C3+' displays 'Space' (4 / 5 GB), 'Charge' (100%), 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom right of the device card.

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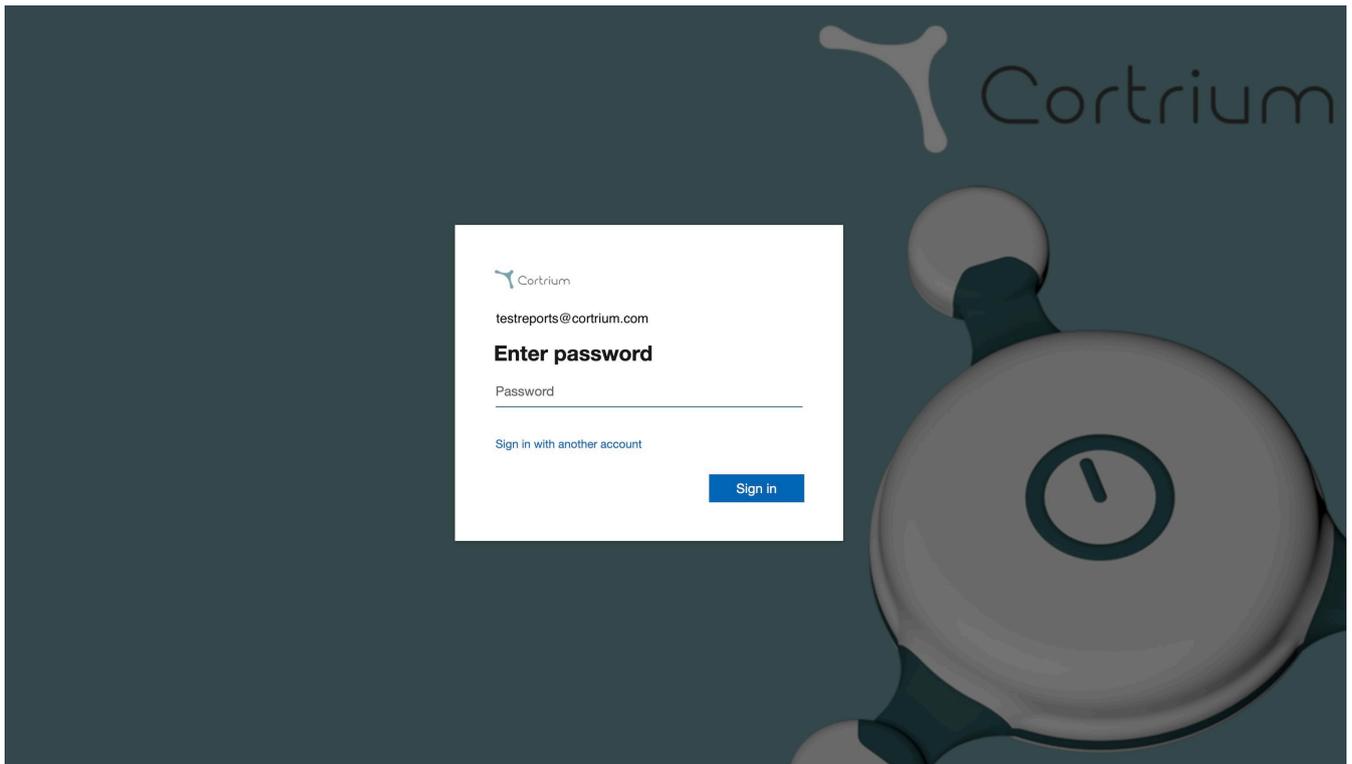
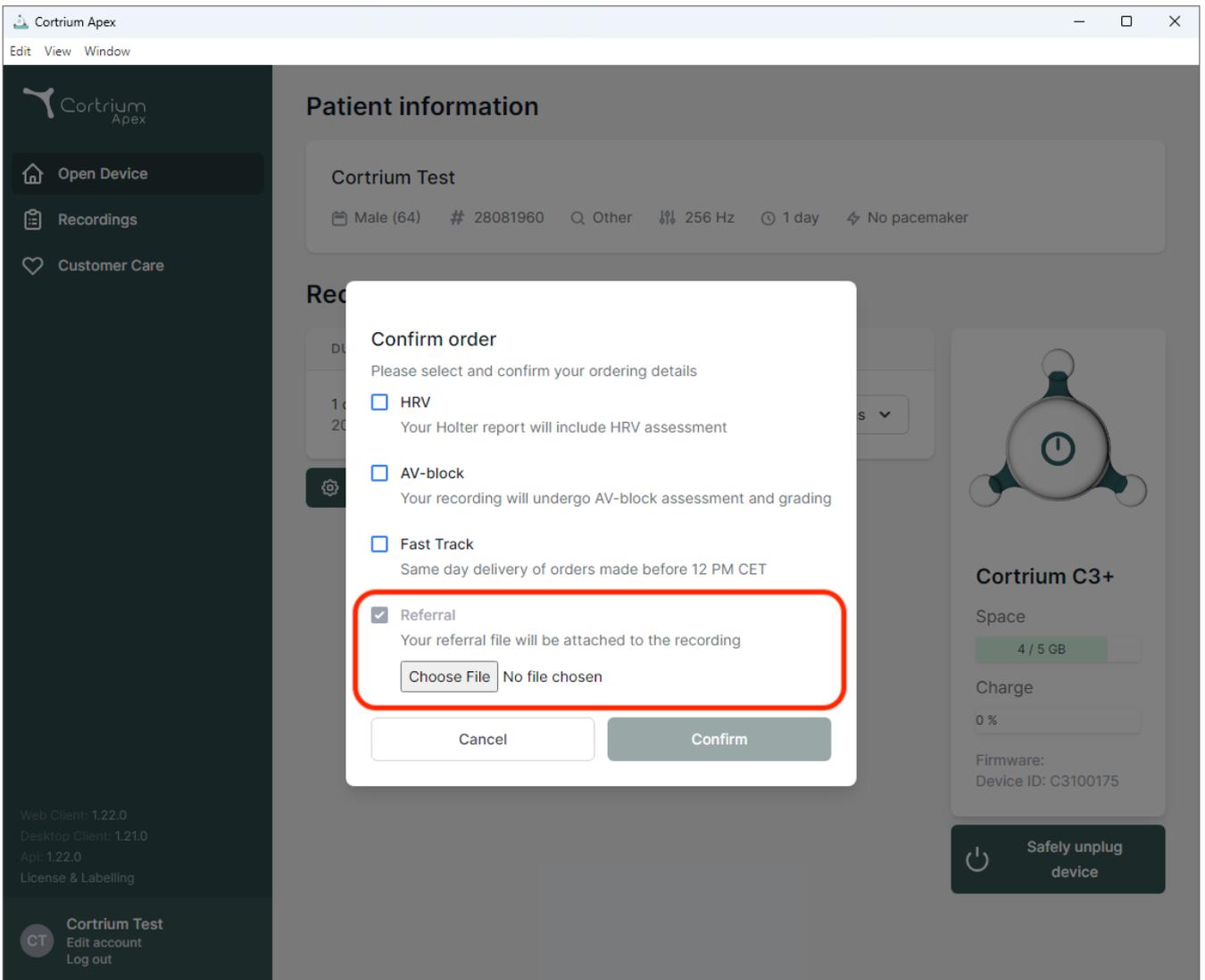
The screenshot displays the Cortrium Apex web application interface. A modal dialog box titled "Confirm order" is centered on the screen, prompting the user to select and confirm their ordering details. The dialog lists several options, each with a checkbox:

- HRV: Your Holter report will include HRV assessment
- AV-block: Your recording will undergo AV-block assessment and grading
- Fast Track: Same day delivery of orders made before 12 PM CET
- Referral: Your referral file will be attached to the recording

The "Referral" option is highlighted with a red border. Below it, there is a "Choose File" button and the text "No file chosen". At the bottom of the dialog are "Cancel" and "Confirm" buttons.


The background interface shows "Patient information" for "Cortrium Test" (Male, 64, # 28081960, 256 Hz, 1 day, No pacemaker). On the right, there is a "Cortrium C3+" device status panel showing 4/5 GB space, 0% charge, and a "Safely unplug device" button.

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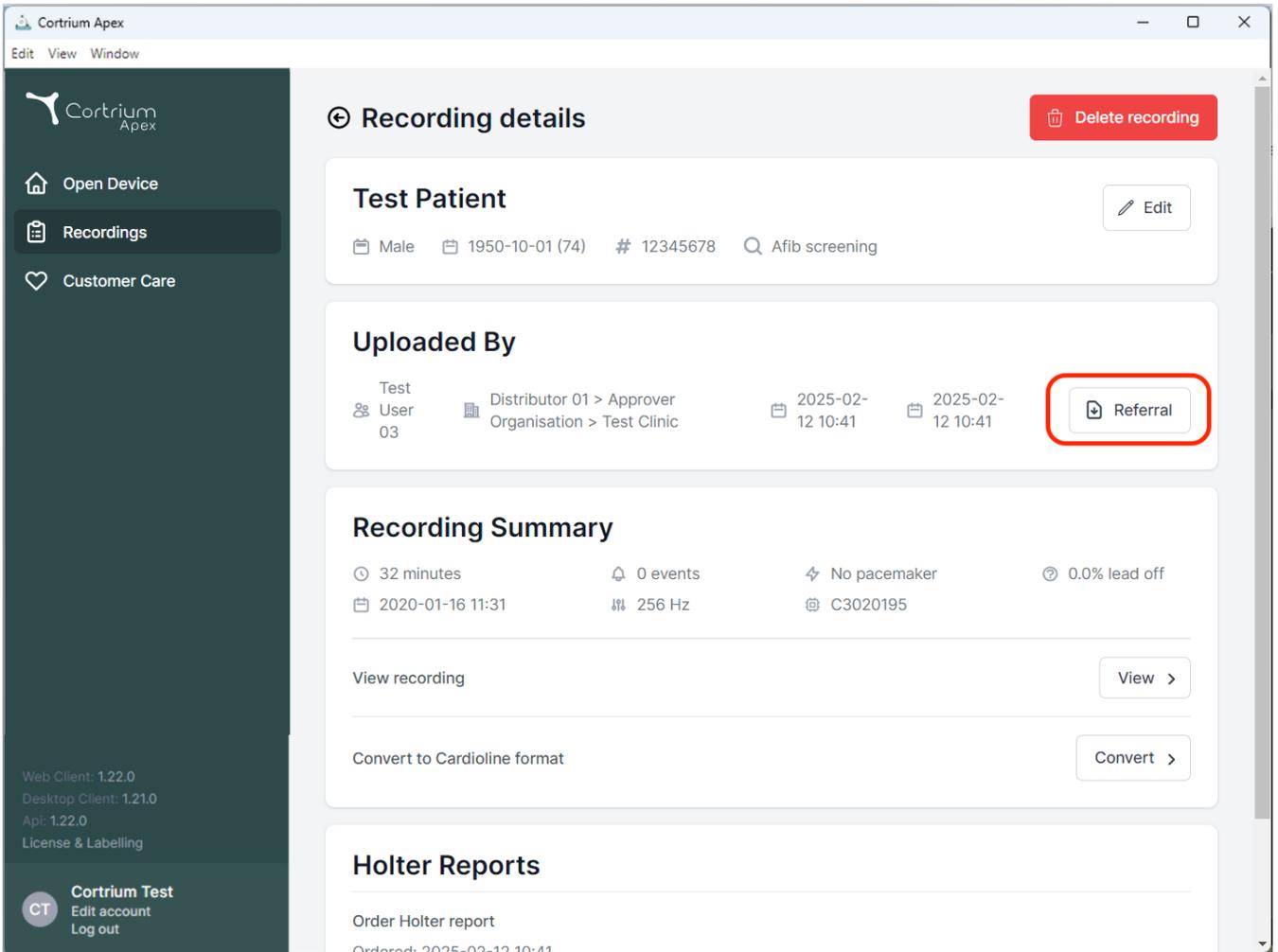The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right. Below this, the 'Test Patient' section shows details for a male patient born on 1950-10-01 (74 years old), with ID # 12345678 and a search term 'Afib screening'. An 'Edit' button is available for this section. The 'Uploaded By' section identifies the user as 'Test User 03' from 'Distributor 01 > Approver Organisation > Test Clinic', with upload dates of 2025-02-12 10:41. A 'Referral' button is highlighted with a red box. The 'Recording Summary' section provides key metrics: 32 minutes duration, 0 events, no pacemaker, and 0.0% lead off. It also shows the recording date (2020-01-16 11:31) and frequency (256 Hz). Below the summary are 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section at the bottom includes an 'Order Holter report' button.

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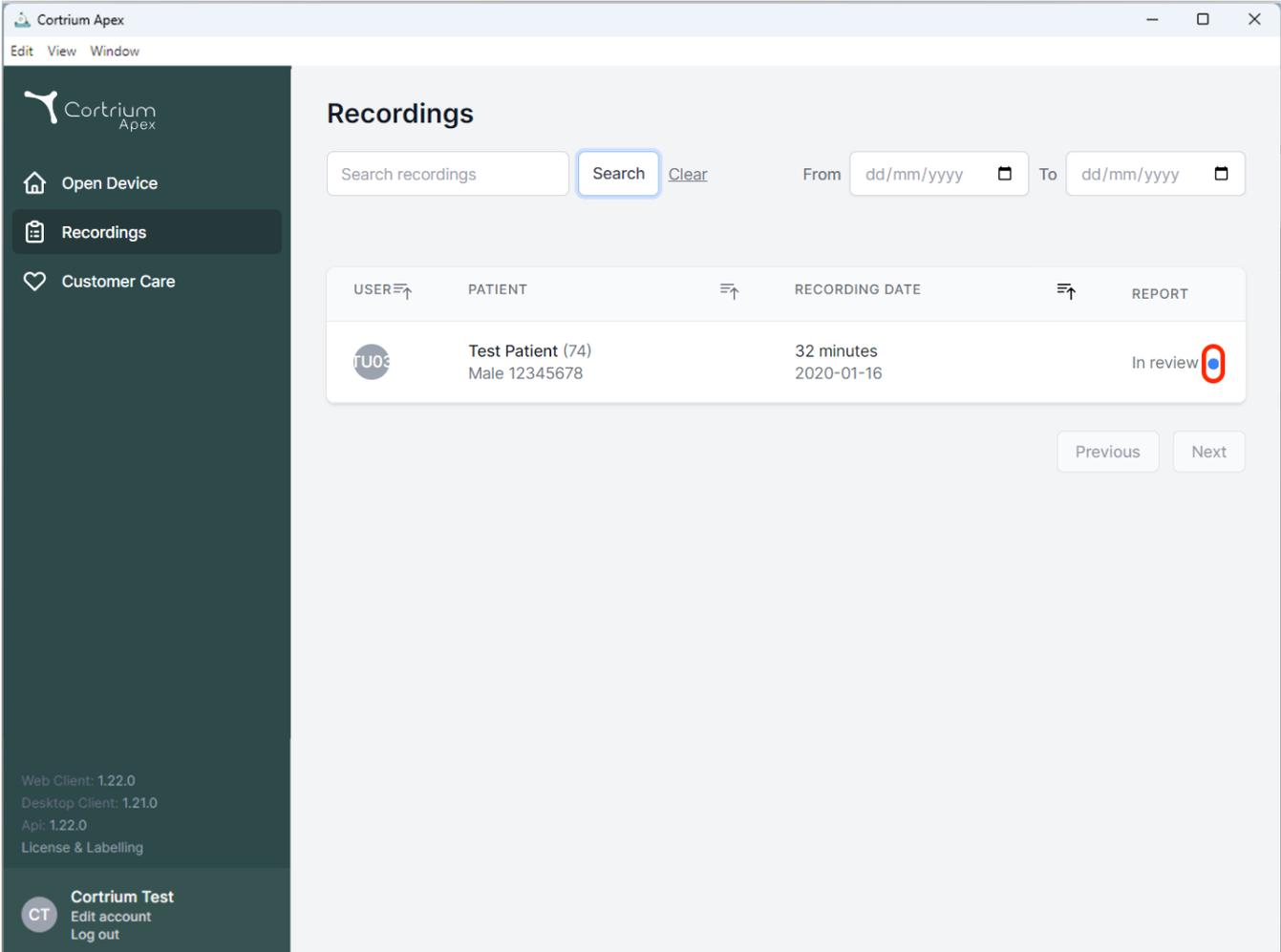
The screenshot displays the 'Recordings' section of the Cortrium Apex application. The interface includes a search bar with a 'Search' button and a 'Clear' link. Below the search bar is a table with the following columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for 'Test Patient (74)' with a recording date of '2020-01-16' and a status of 'In review'. A blue dot is visible next to the 'In review' status. The sidebar on the left contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The bottom of the sidebar shows version information and a user profile for 'Cortrium Test'.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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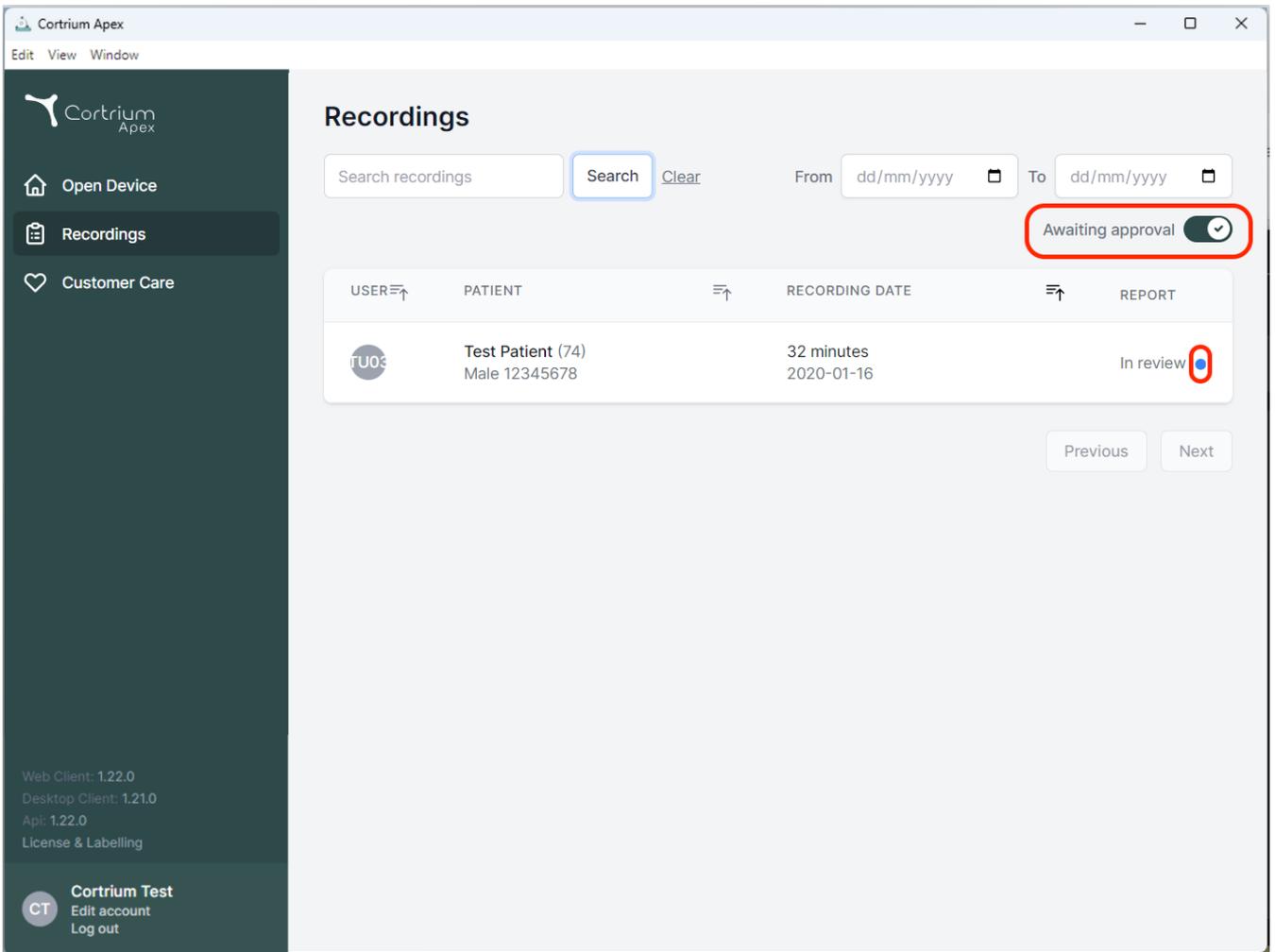
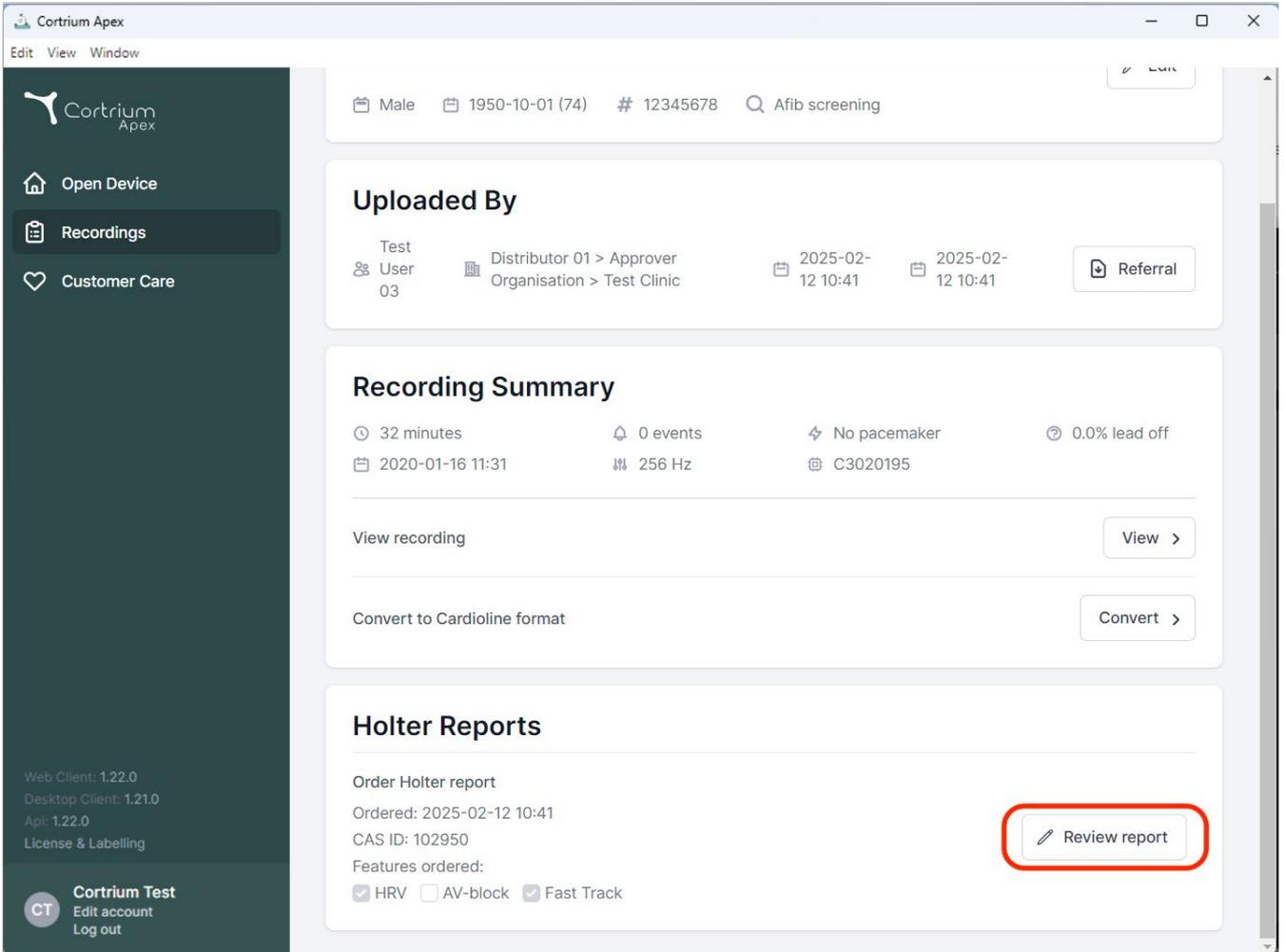
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links for 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Uploaded By:** Shows test details including 'Test User 03', 'Distributor 01 > Approver Organisation > Test Clinic', and dates '2025-02-12 10:41'. A 'Referral' button is present.
- Recording Summary:** Displays recording statistics: '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off'. It also shows the recording date '2020-01-16 11:31', '256 Hz', and 'C3020195'. Action buttons for 'View recording' and 'Convert to Cardioline format' are available.
- Holter Reports:** Shows 'Order Holter report' details: 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red box.

At the bottom left, the user 'Cortrium Test' is logged in, with options to 'Edit account' and 'Log out'. Version information for Web Client (1.22.0), Desktop Client (1.21.0), and Api (1.22.0) is also visible.

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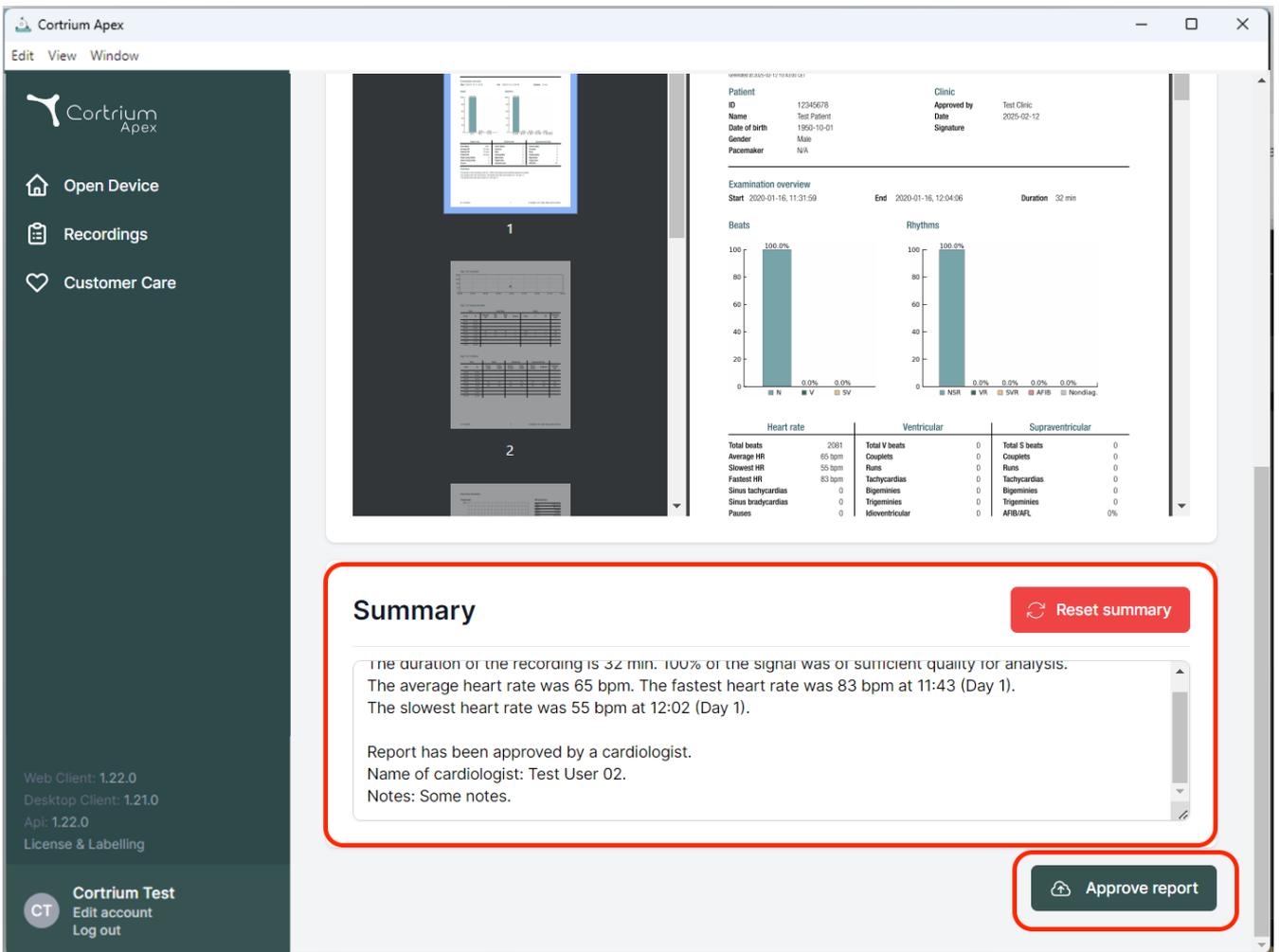The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area shows a patient report for ID 12345678, including patient details, examination overview (start/end times and duration), and two bar charts for Beats and Rhythms. Below the charts is a table with three columns: Heart rate, Ventricular, and Supraventricular, containing various cardiac metrics. At the bottom, a red-bordered box highlights the 'Summary' section, which contains a 'Reset summary' button and a text area with the following text: 'The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1). Report has been approved by a cardiologist. Name of cardiologist: Test User 02. Notes: Some notes.' To the right of the summary box is an 'Approve report' button.

02/2025

Cortrium ApS · Erik Husfeldts Vej 7 · DK-2630 Taastrup · Dinamarca  
E-mail: [info@cortrium.com](mailto:info@cortrium.com) Website: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033