Cortrium Apex 1.22

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
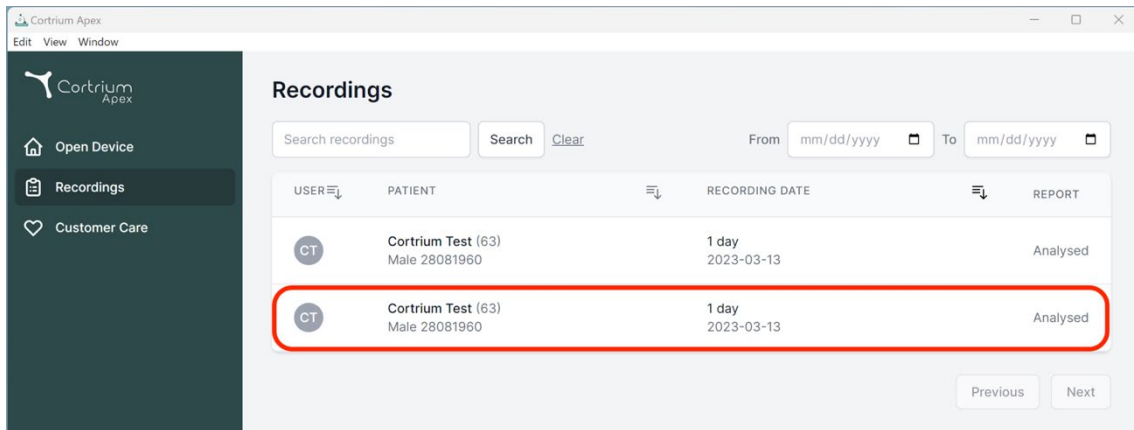
Recordings

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

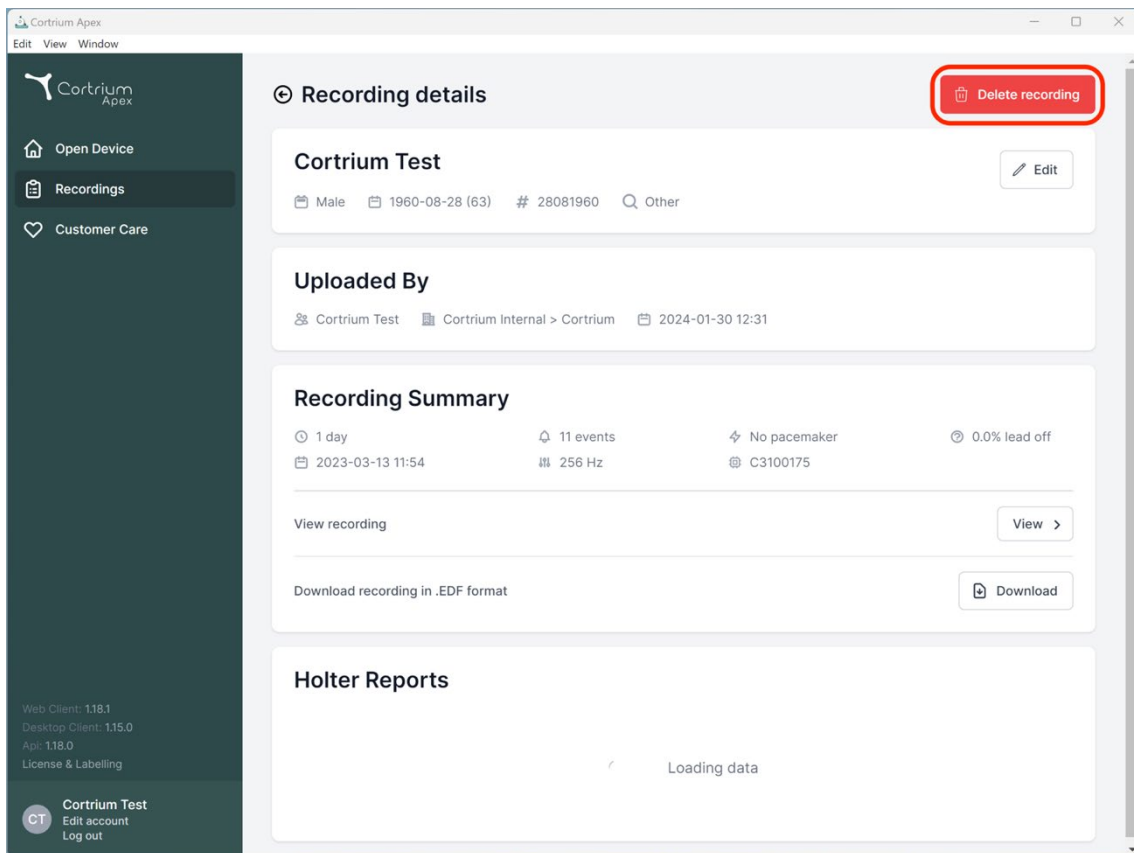Previous Next

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View

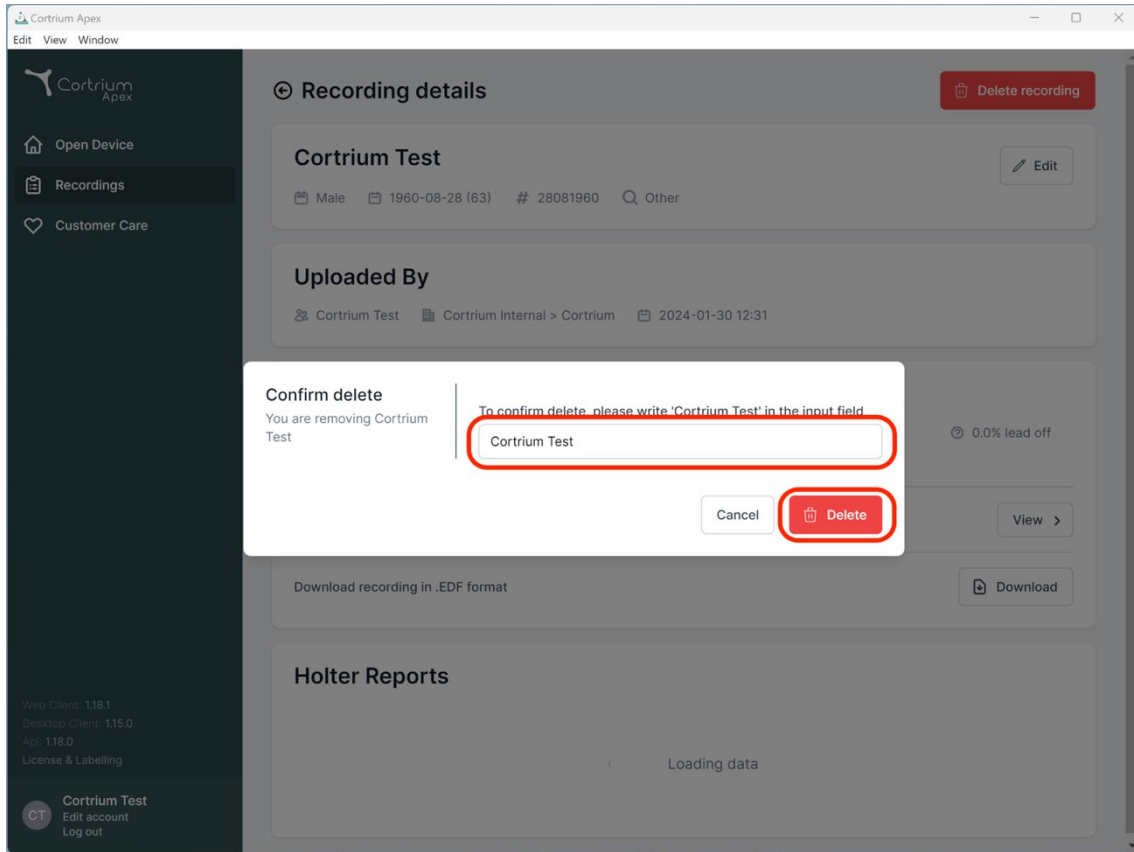
Download recording in .EDF format Download

Holter Reports

Loading data

Web Client: 1.18.1
Desktop Client: 1.15.0
Api: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

Cortrium Test

Cancel Delete

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

0.0% lead off

View >

Download recording in .EDF format Download

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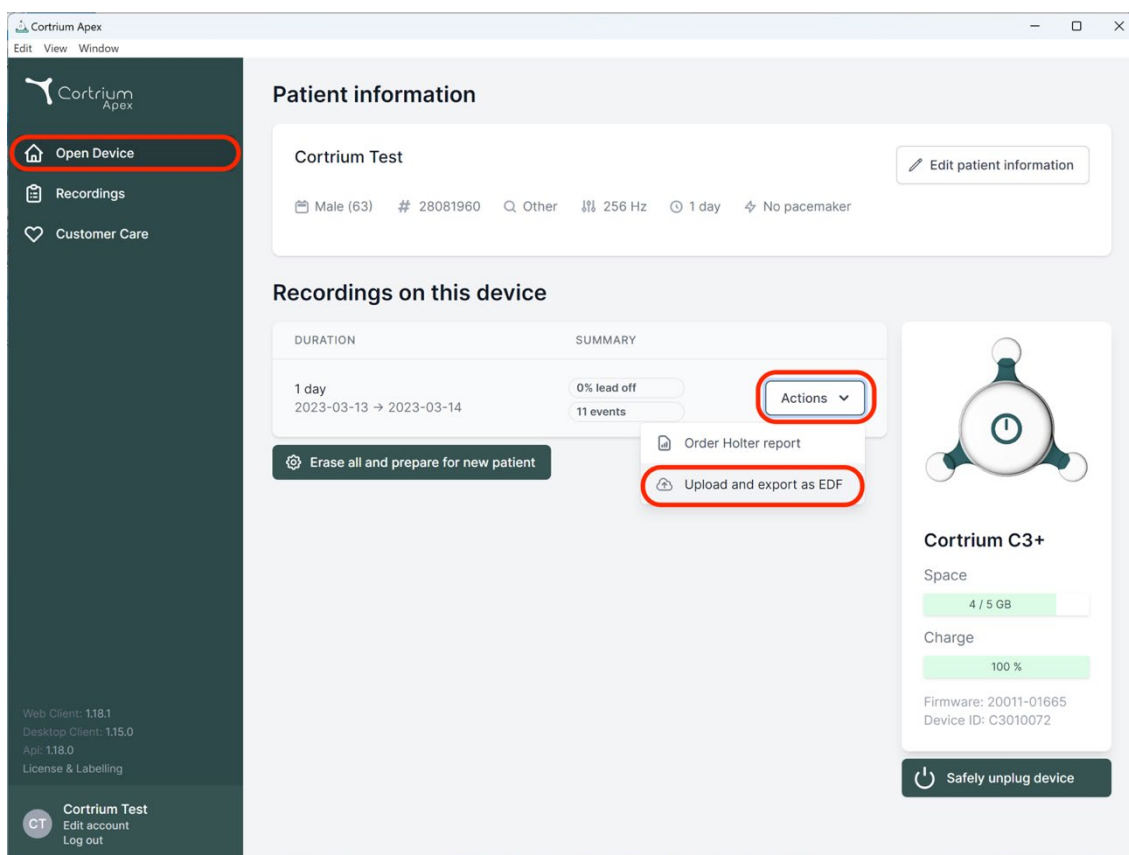
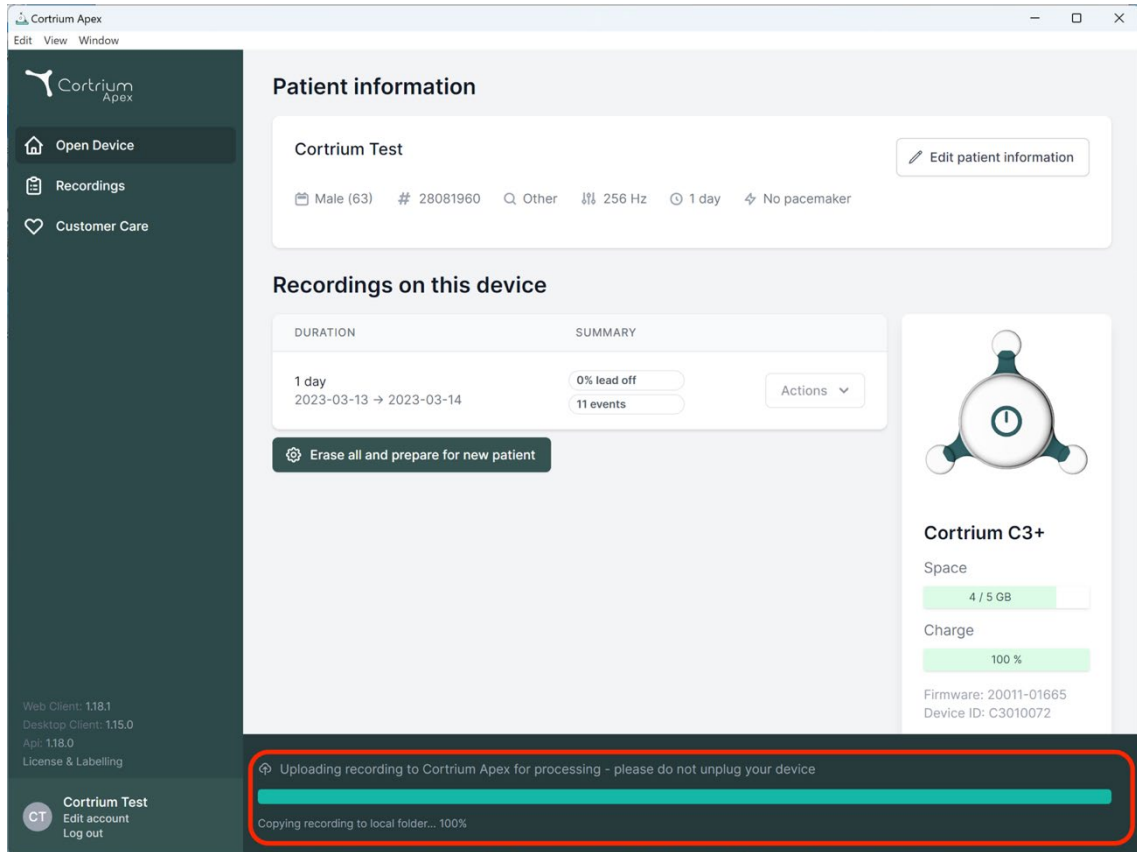
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

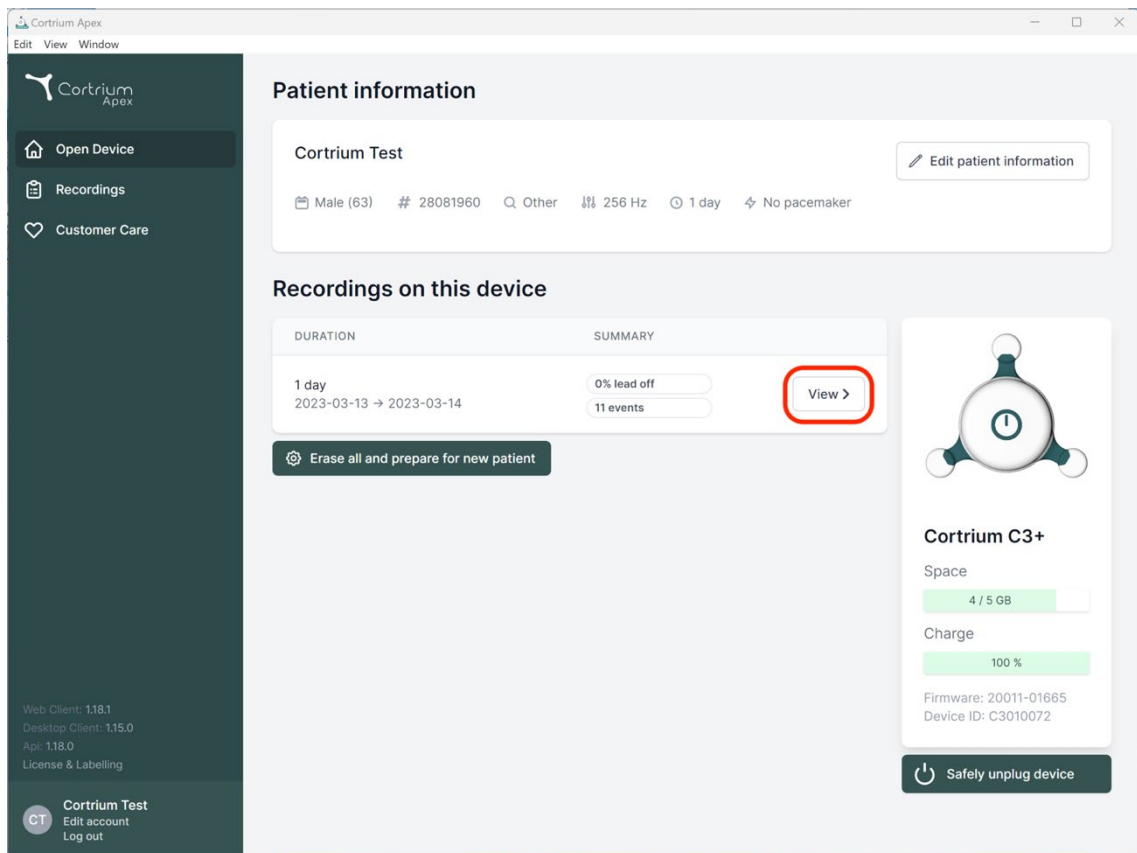
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
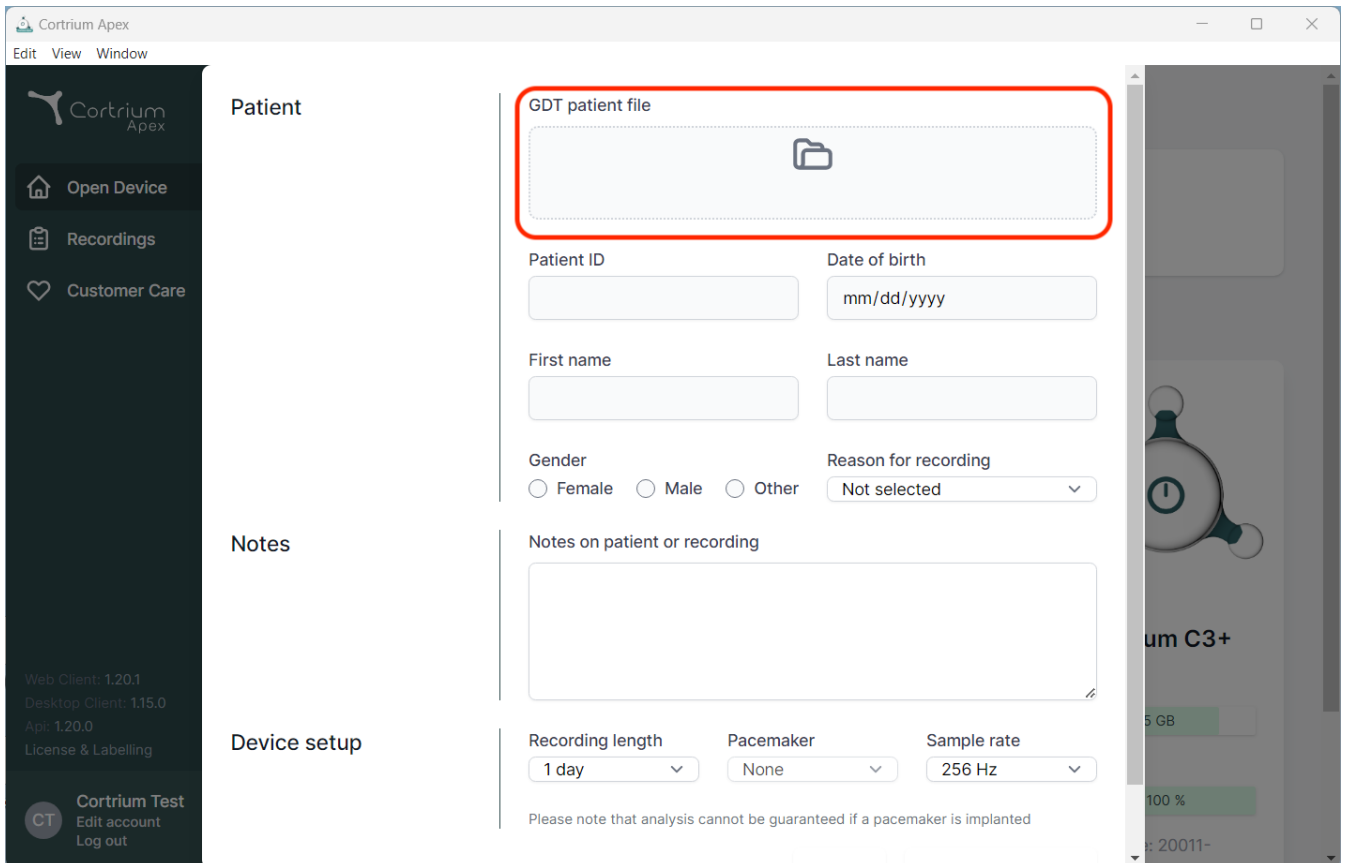
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details for the Web Client (1.20.1), Desktop Client (1.15.0), and API (1.20.0), along with a 'License & Labelling' link. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload field, which is highlighted with a red rectangle and contains a folder icon. Below this are input fields for 'Patient ID', 'Date of birth' (with a 'mm/dd/yyyy' placeholder), 'First name', and 'Last name'. There are also radio buttons for 'Gender' (Female, Male, Other) and a dropdown for 'Reason for recording' (currently 'Not selected'). The 'Notes' section has a text area labeled 'Notes on patient or recording'. The 'Device setup' section includes dropdowns for 'Recording length' (set to '1 day'), 'Pacemaker' (set to 'None'), and 'Sample rate' (set to '256 Hz'). A disclaimer at the bottom of this section states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. On the right side of the interface, a partial view of another screen shows a device icon and the text 'um C3+'.

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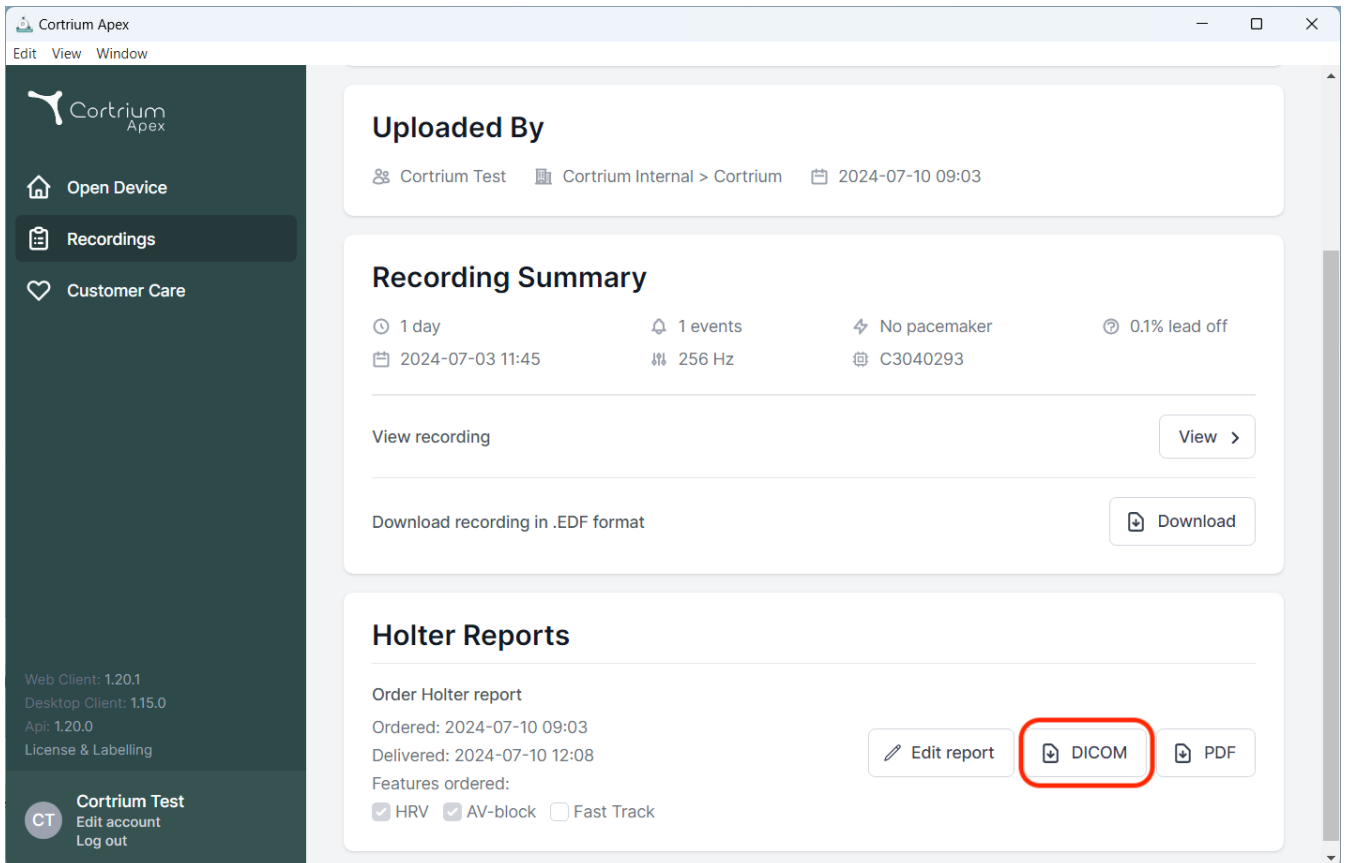
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections. The top section, 'Uploaded By', shows the user 'Cortrium Test' and the upload date '2024-07-10 09:03'. The middle section, 'Recording Summary', provides details about the recording: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'. The bottom section, 'Holter Reports', shows the report order details: 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists ordered features: 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked). At the bottom right of this section are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

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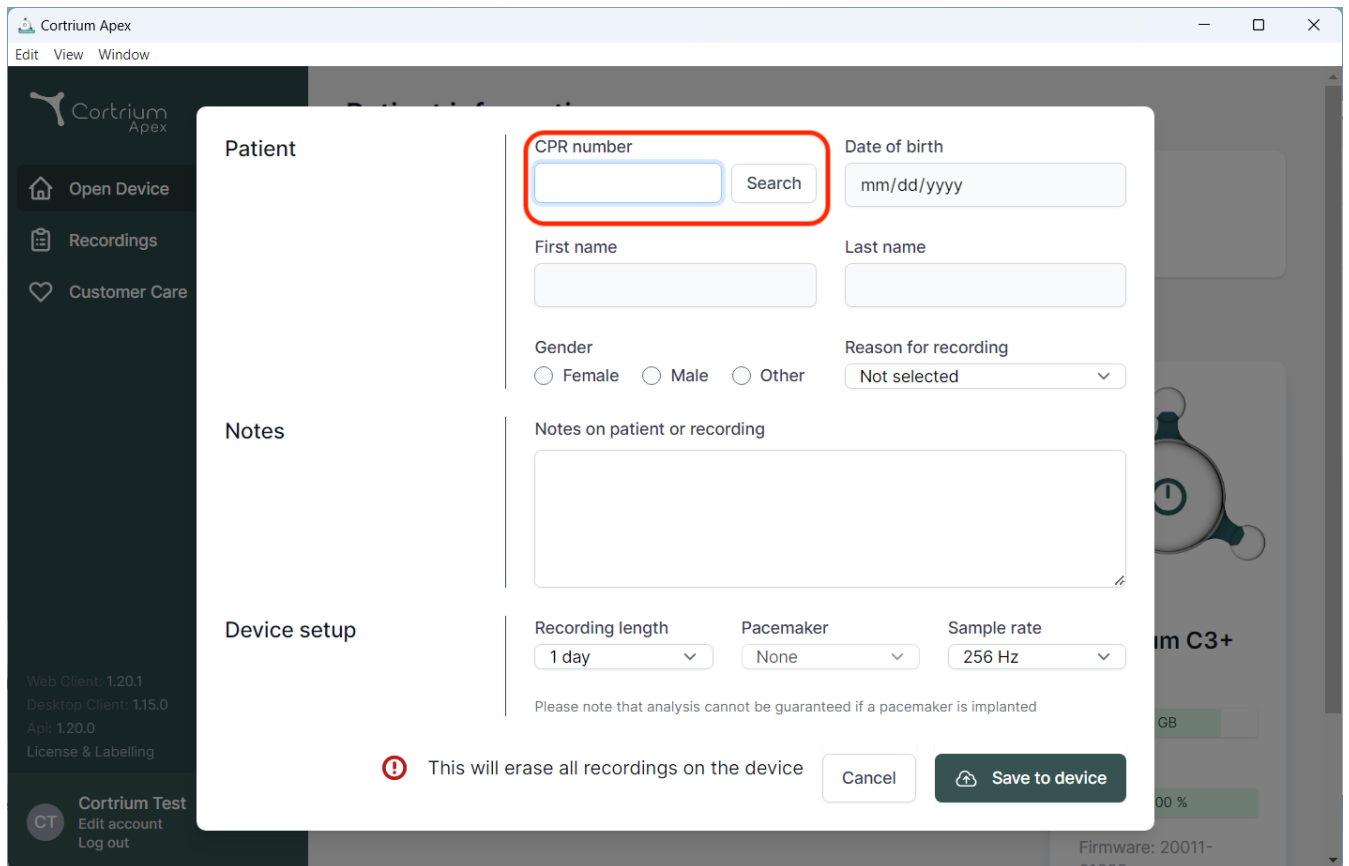Notes on patient or recording

Device setup

Recording length Pacemaker Sample rate

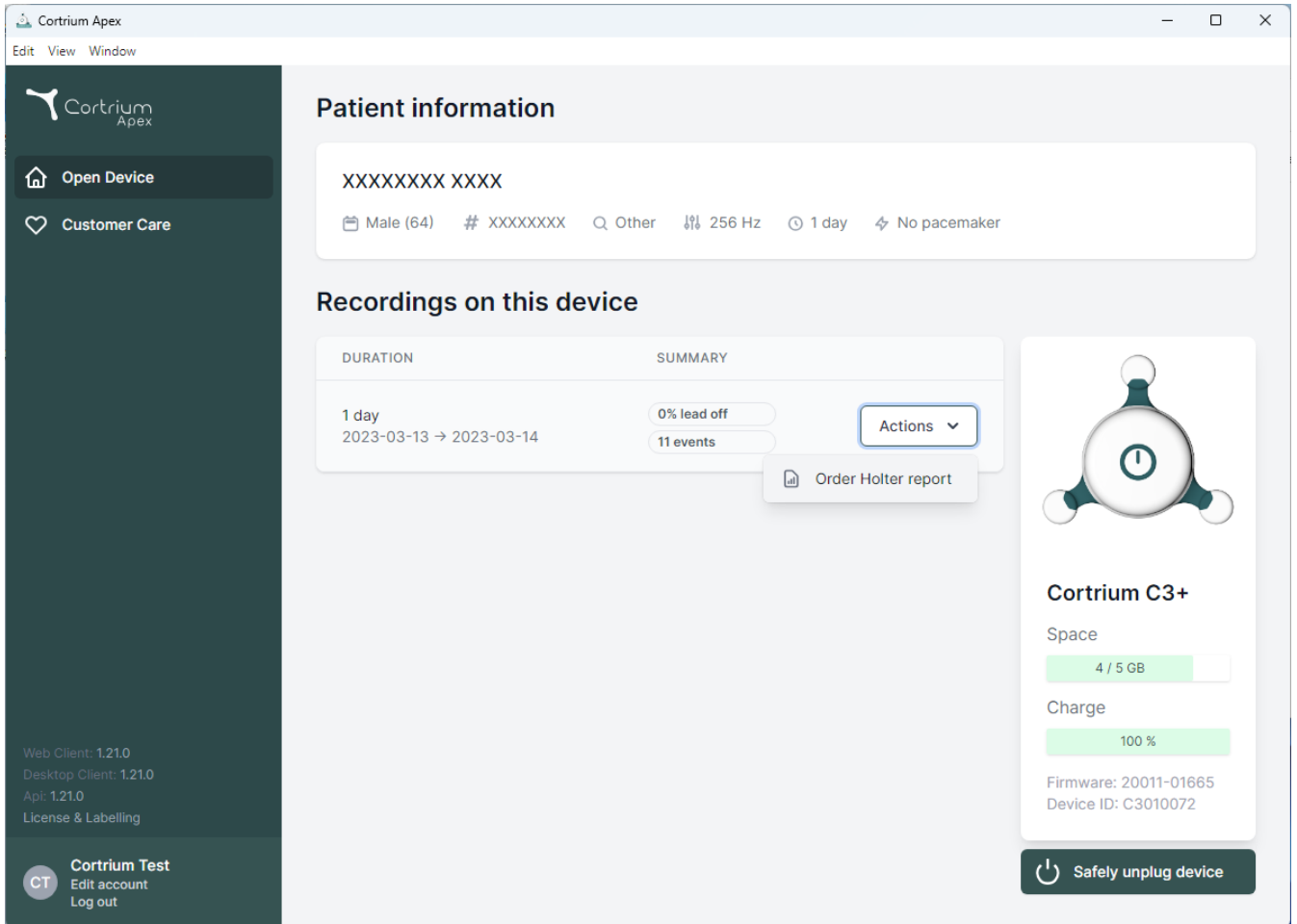
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device' and 'Customer Care'. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

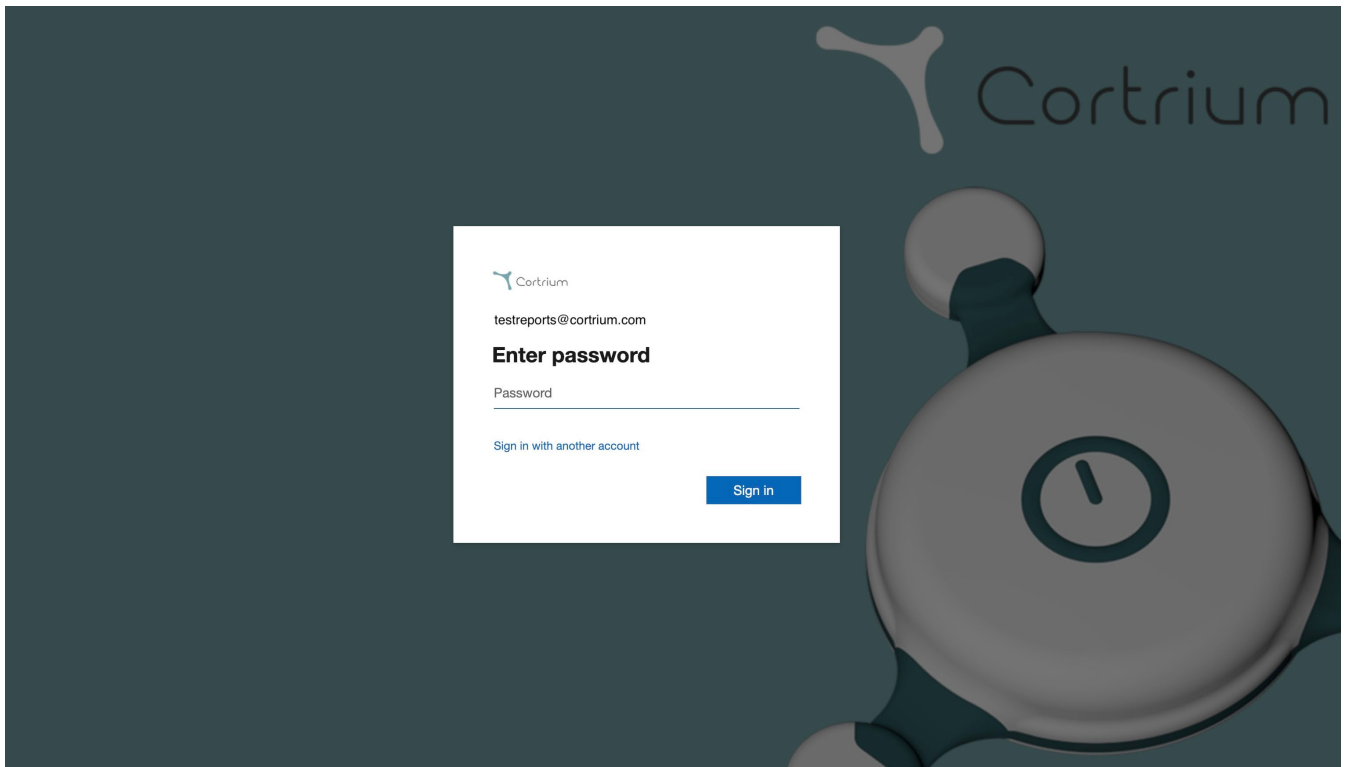
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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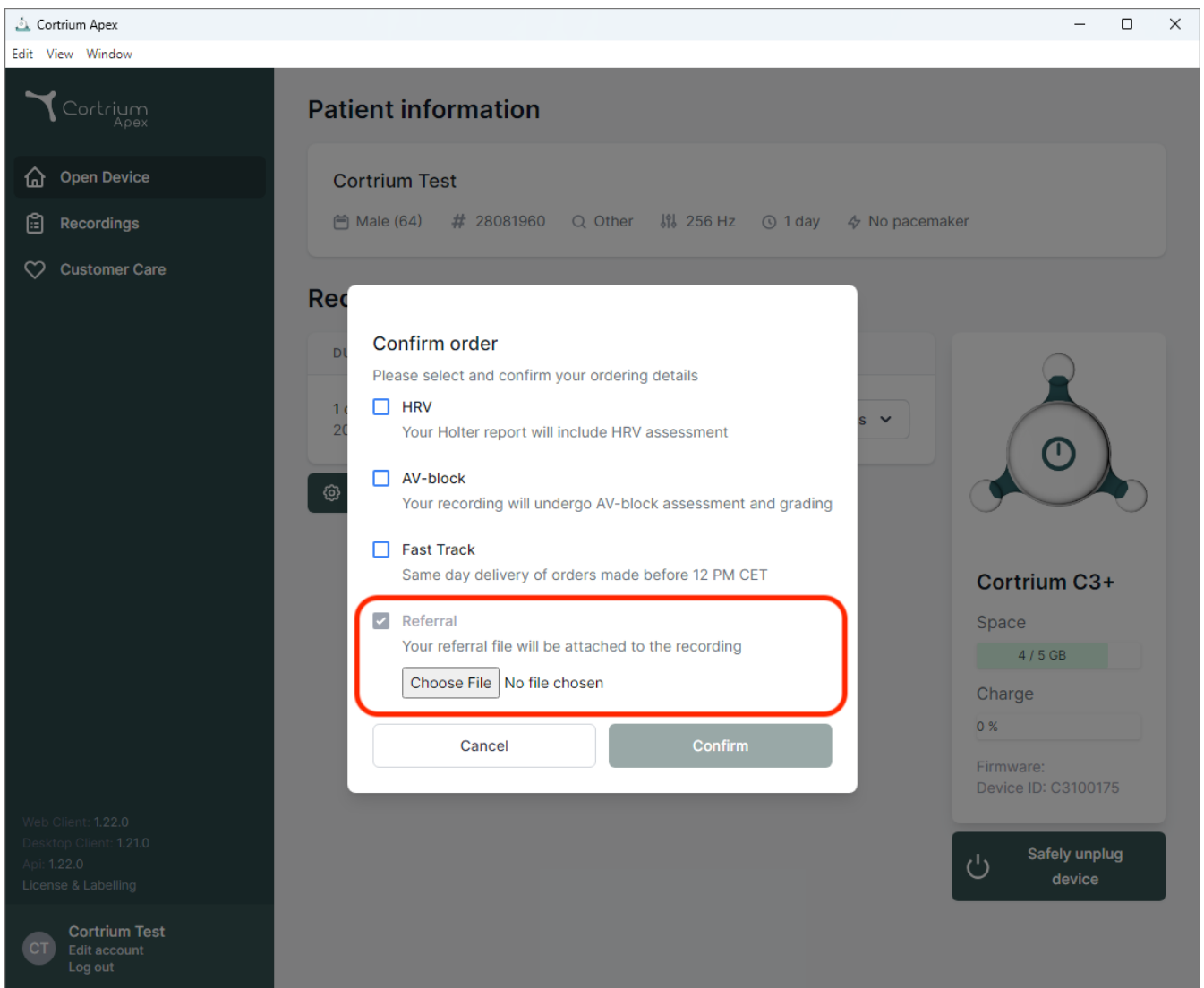
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
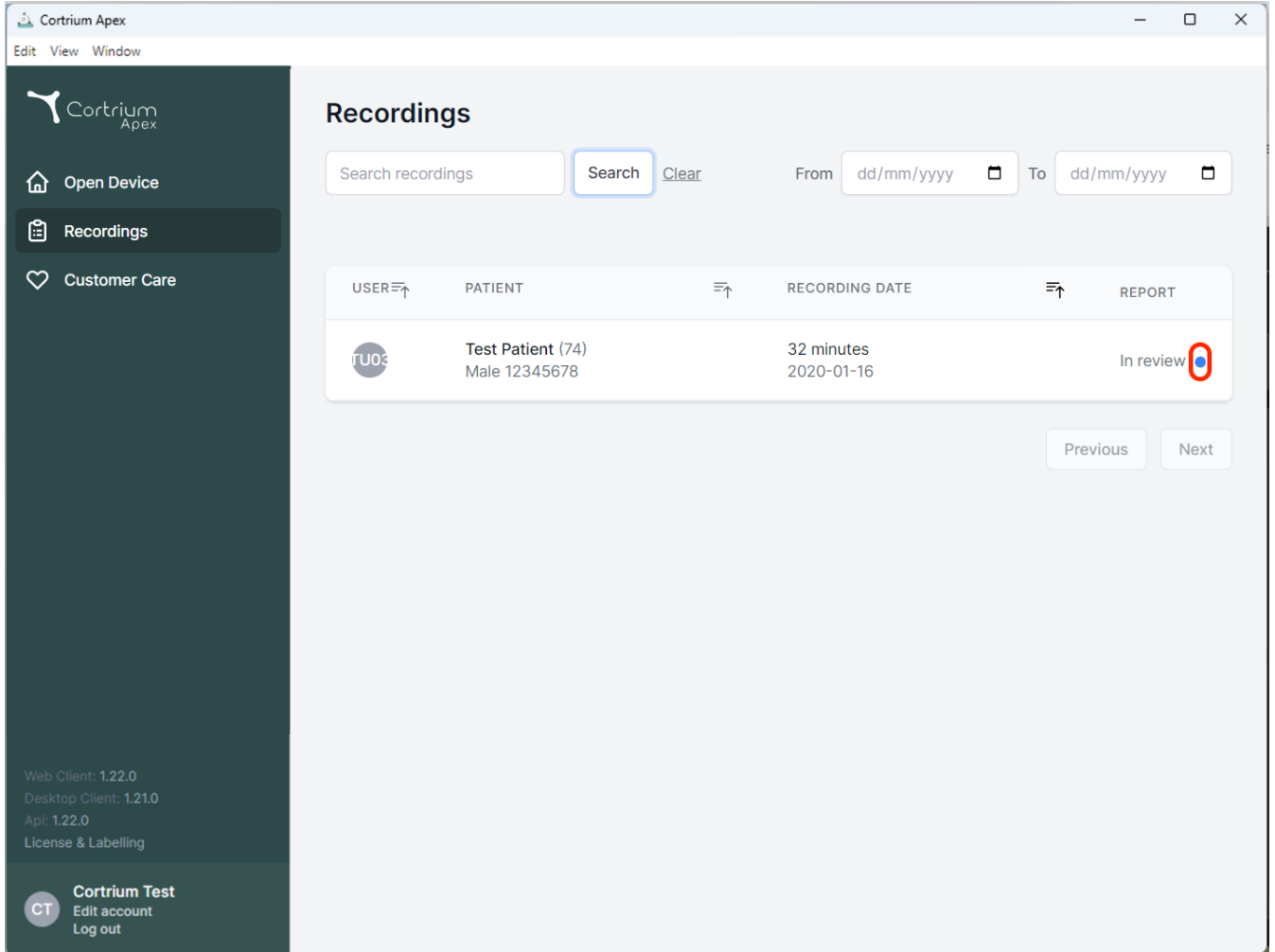
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (selected), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot is visible next to the "In review" status. At the bottom of the table are "Previous" and "Next" buttons. The footer of the sidebar displays version information and a user profile for "Cortrium Test".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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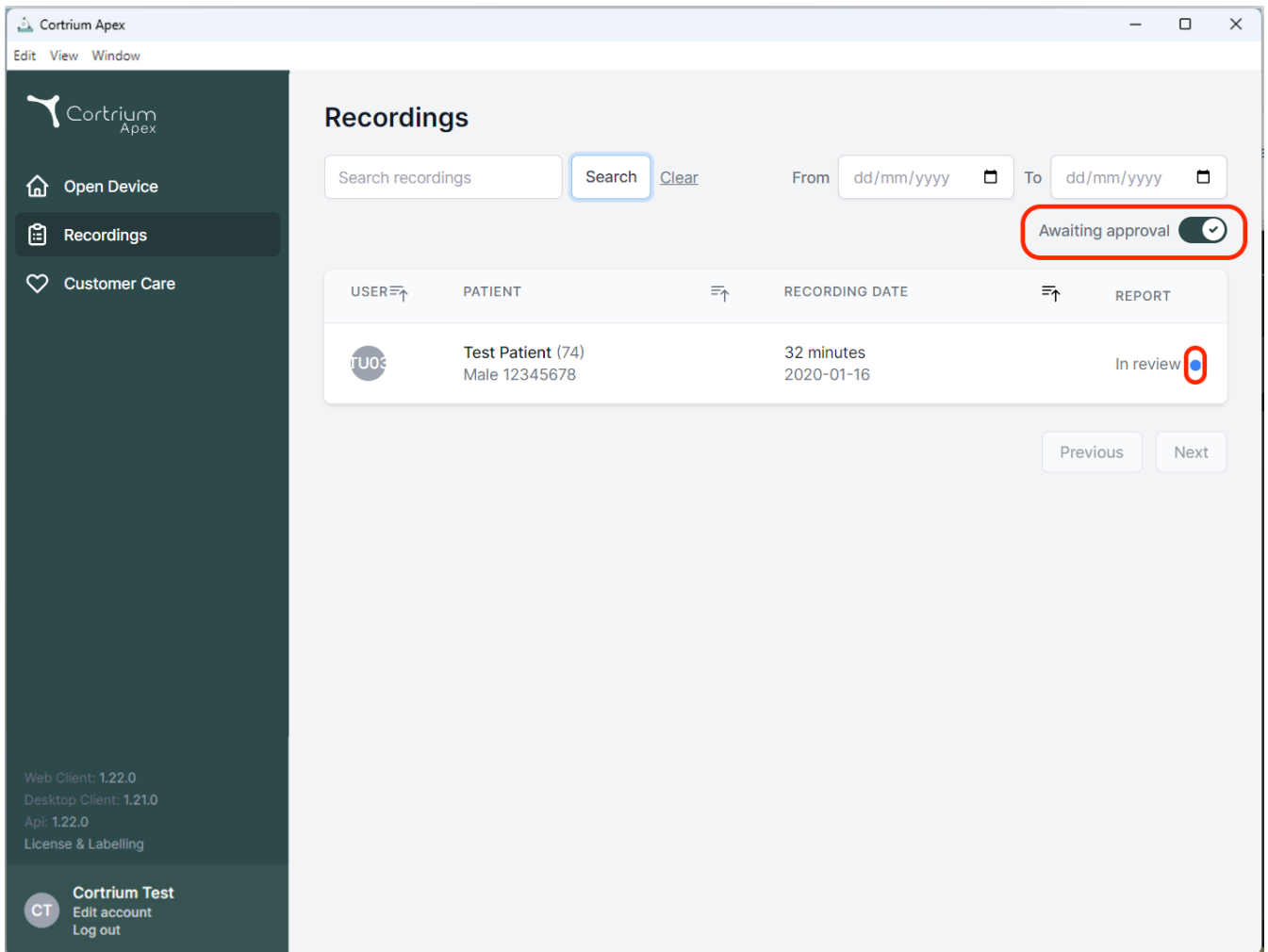
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings (selected), and Customer Care. The main area is titled 'Recordings' and contains a search bar, date filters, and a table of recordings. A red box highlights the 'Awaiting approval' toggle switch, which is currently turned on. Below this, a table lists recordings, with one entry for 'Test Patient (74)' showing a status of 'In review' and a blue dot icon. At the bottom left of the sidebar, version information and a 'Cortrium Test' button are visible.


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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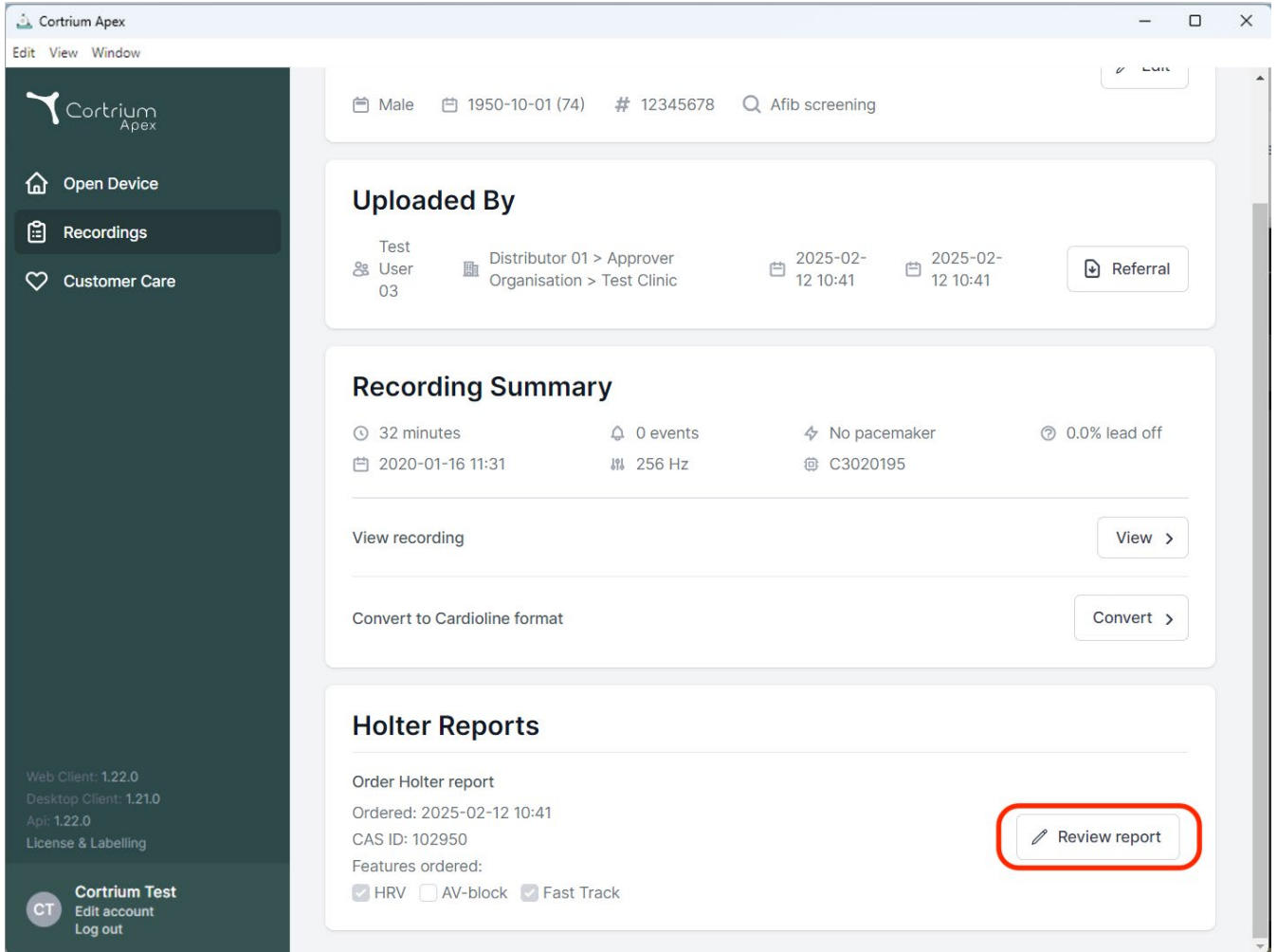The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. Below these are version details (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area shows patient details (Male, 1950-10-01, #12345678, Afib screening) and a 'Referral' button. The 'Uploaded By' section lists 'User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates. The 'Recording Summary' section provides details like '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off', along with a 'View recording' button. The 'Holter Reports' section shows an 'Order Holter report' with a 'Review report' button highlighted by a red rectangle. Below this, it lists the ordered date, CAS ID, and features ordered (HRV, AV-block, Fast Track).

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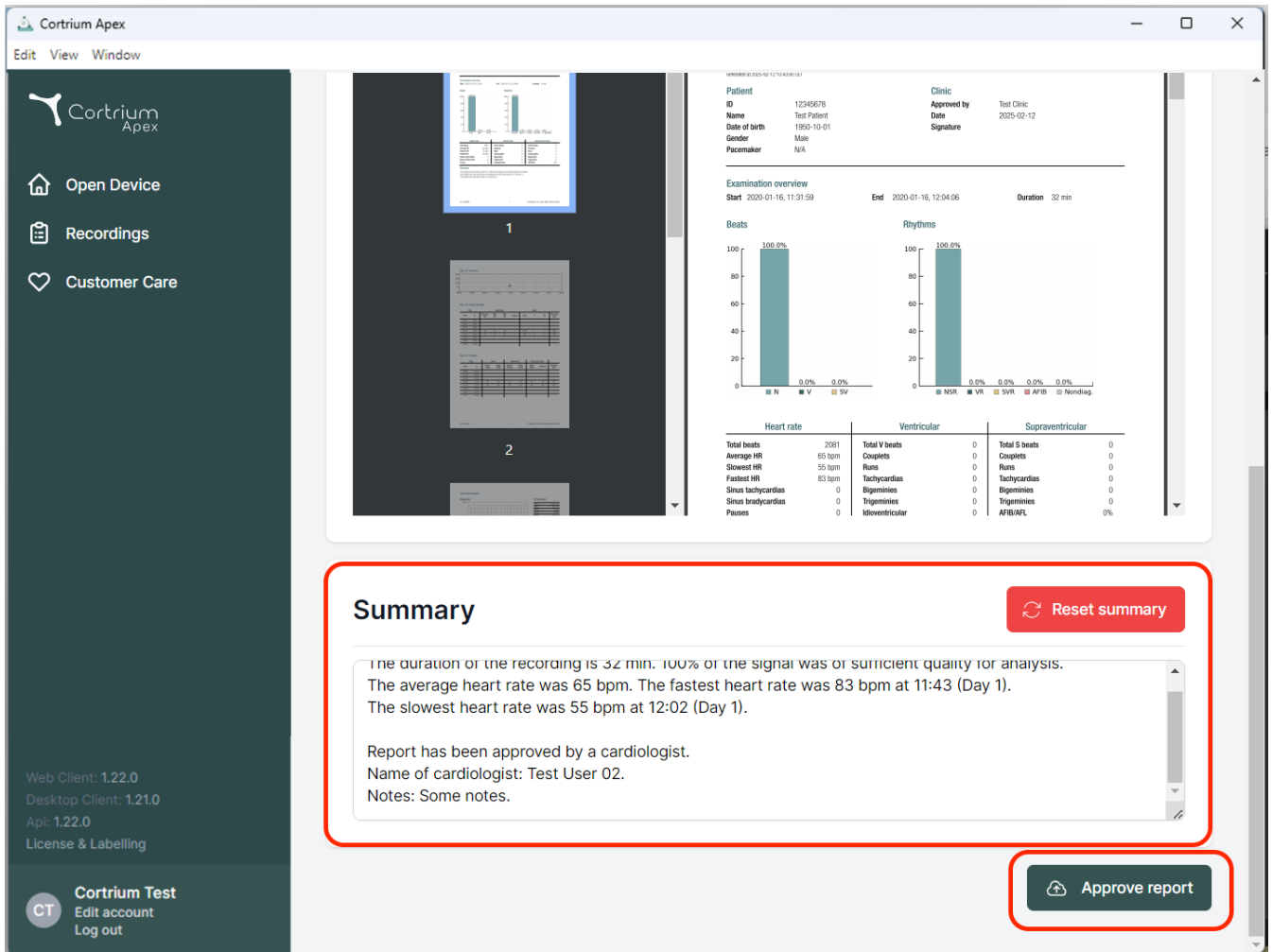Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

Beats
100.0%
0.0%
0.0%

Rhythms
100.0%
0.0%
0.0%
0.0%
0.0%
0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AF/IB/ATL	0%

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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