Cortrium Apex 1.22

Bruksanvisning

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Order Holter report
Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00
Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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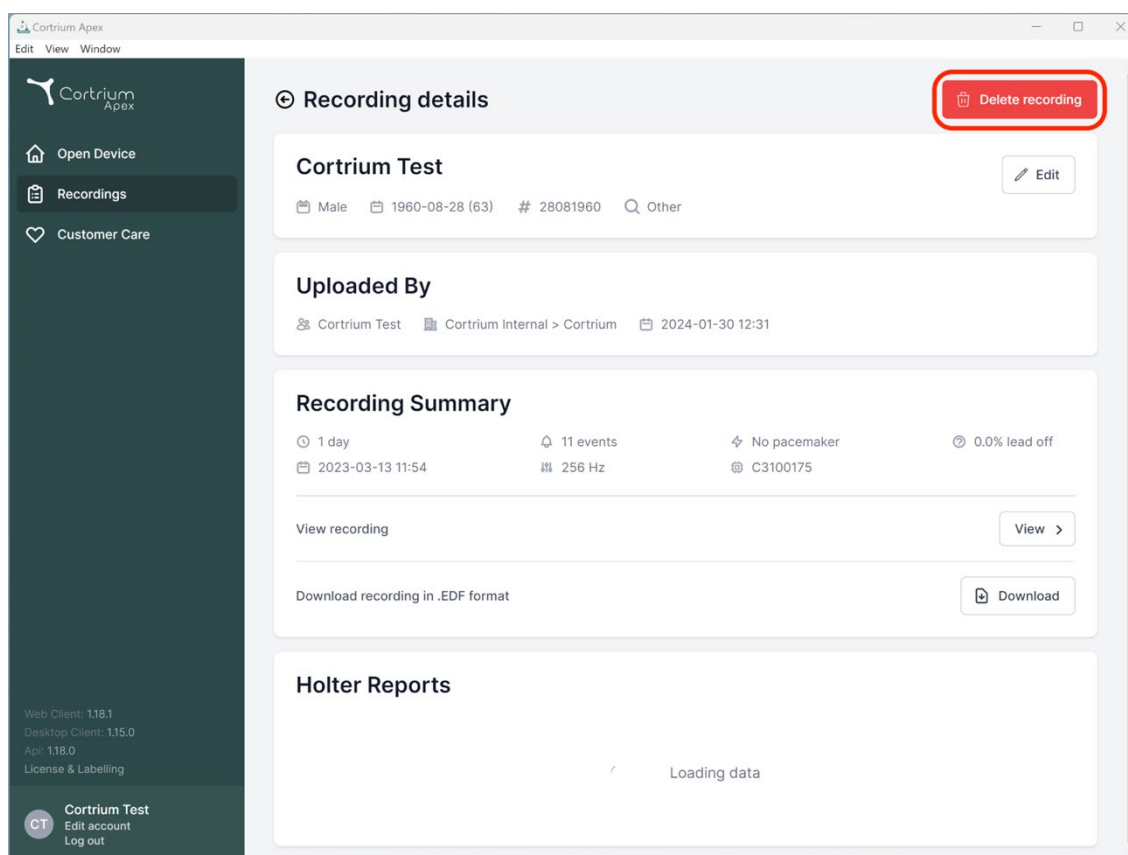
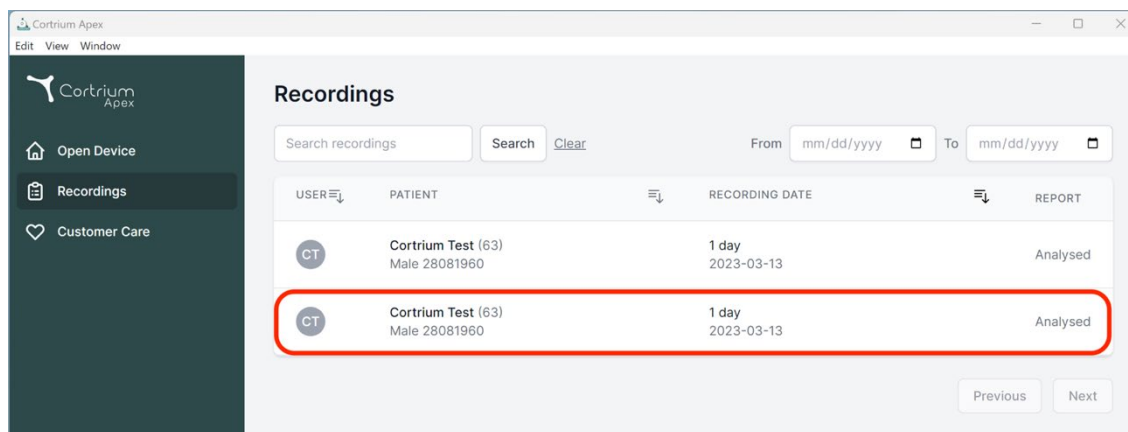
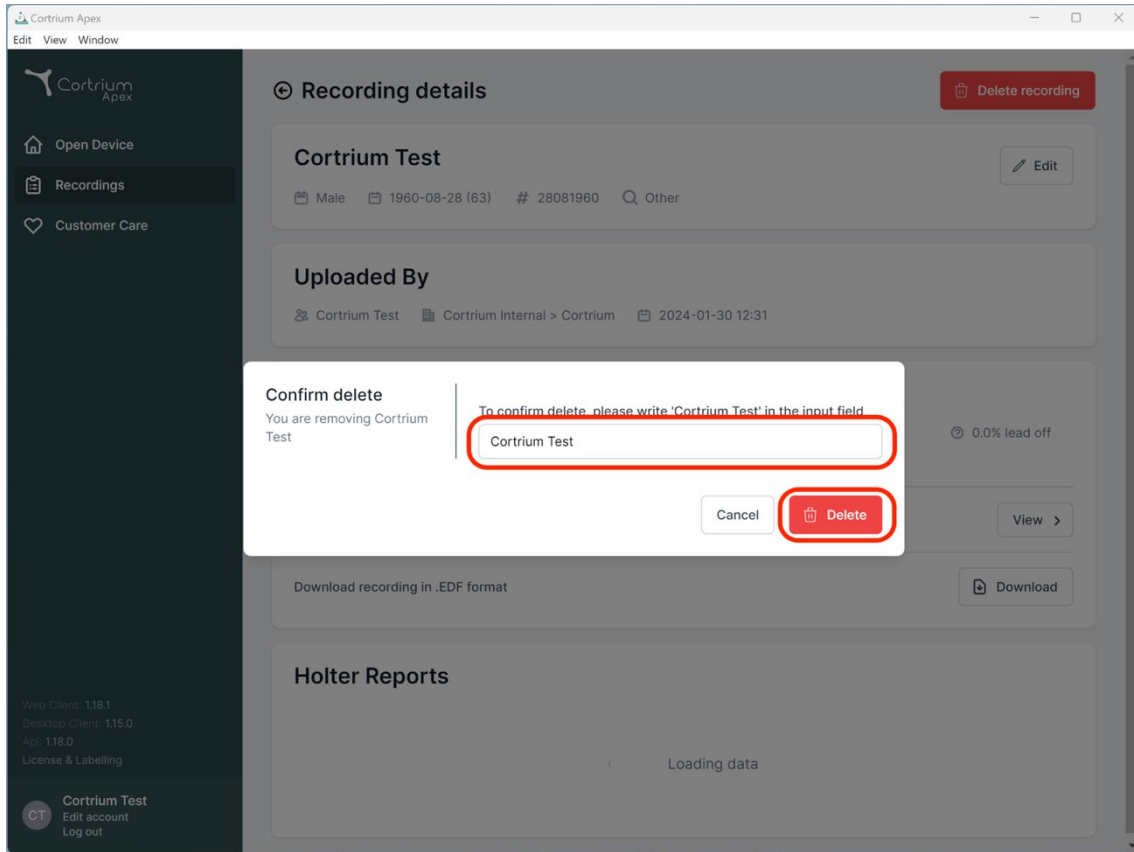
Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

Cortrium Test

Cancel Delete

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

0.0% lead off

View >

Download recording in .EDF format Download

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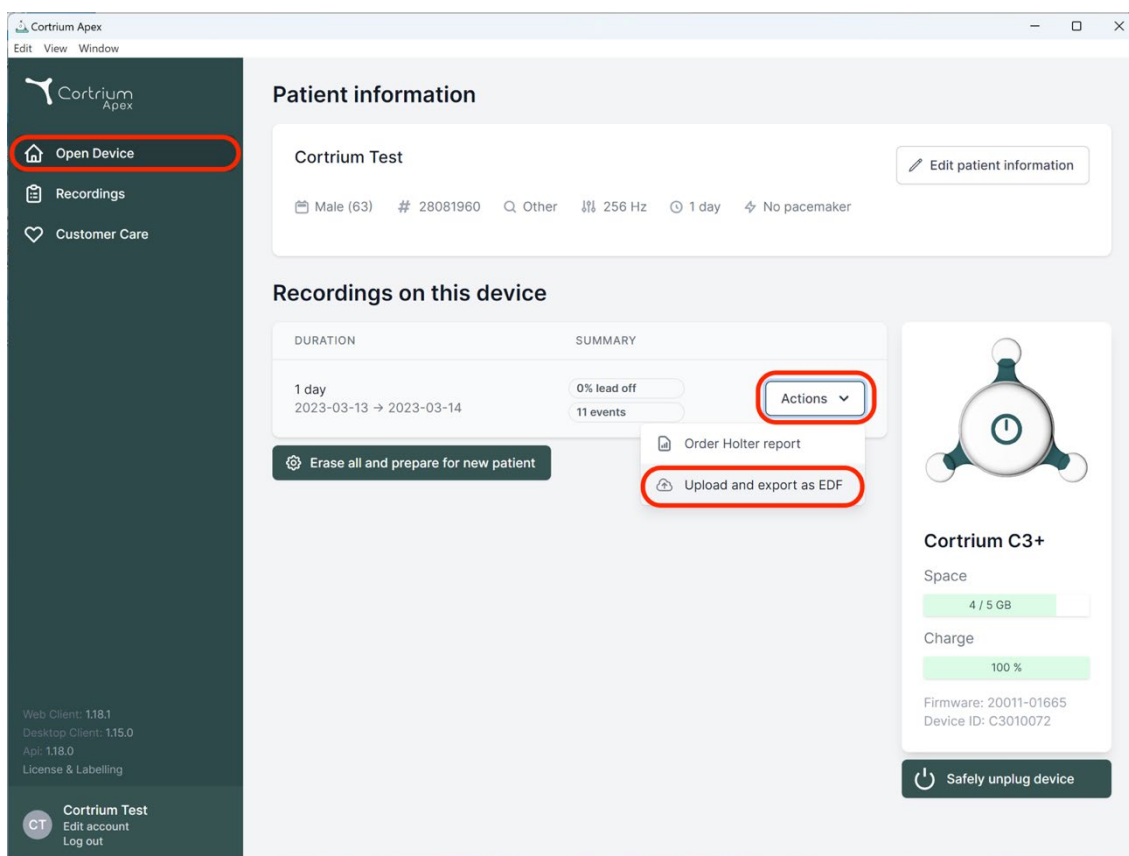
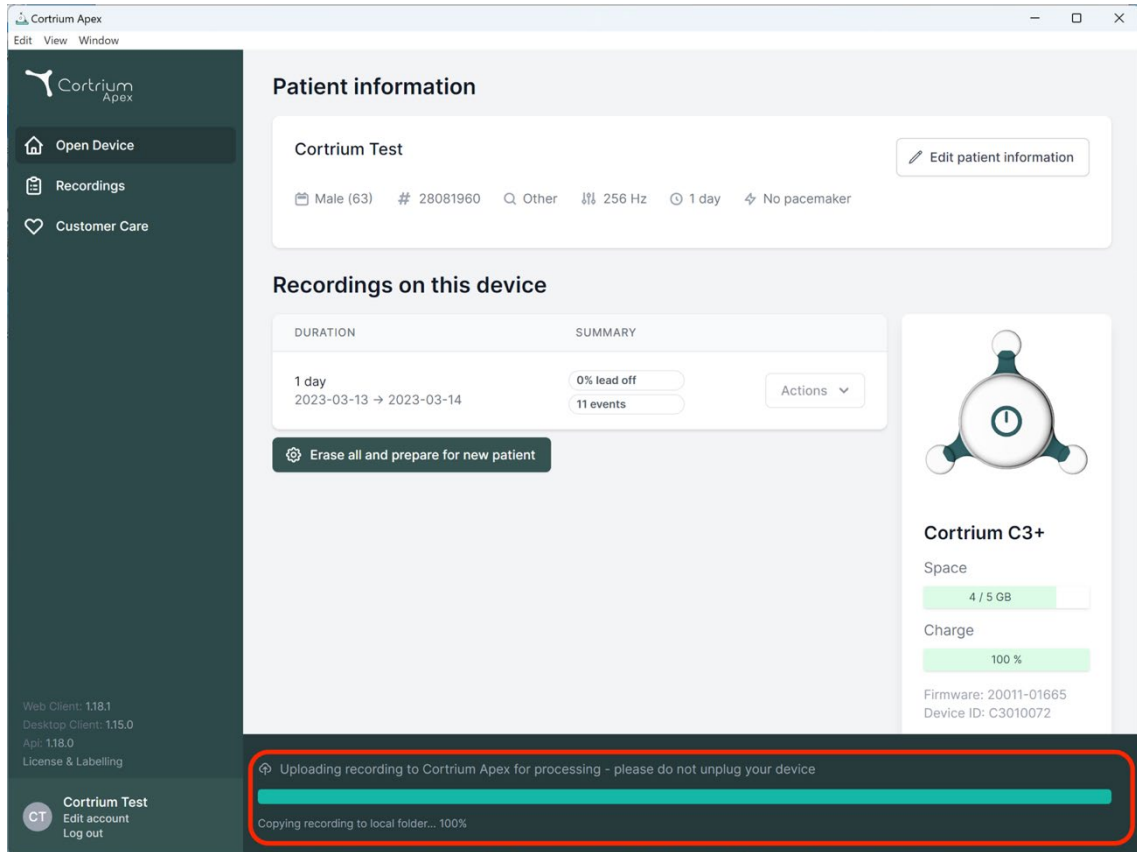
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

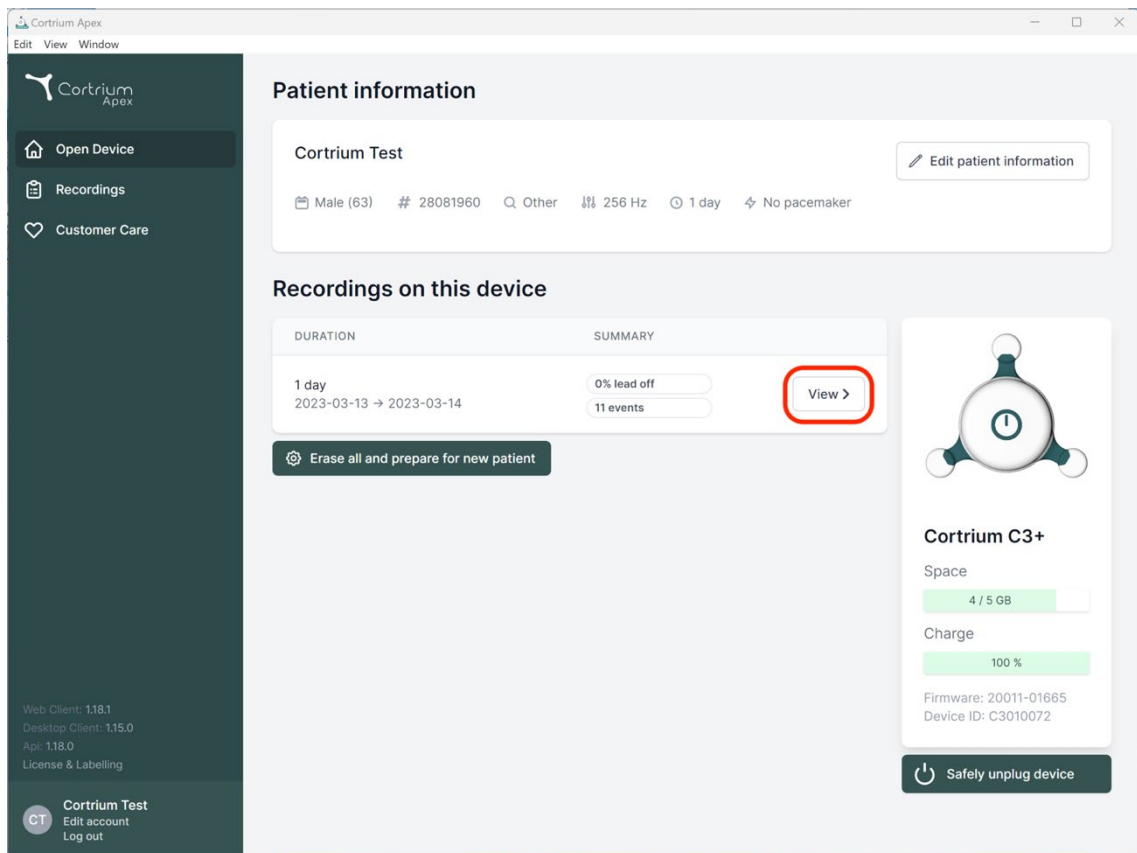
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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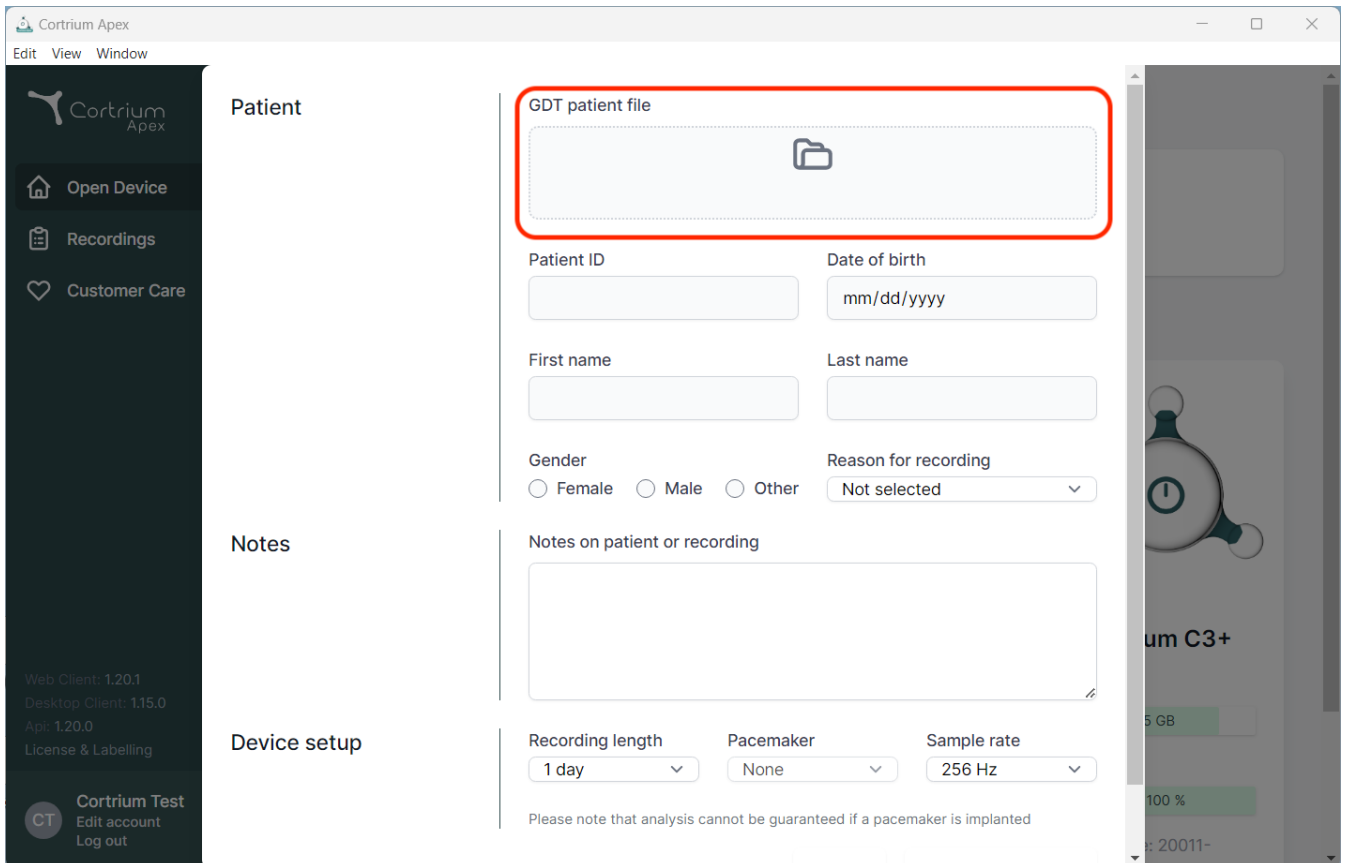
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The 'GDT patient file' upload area is highlighted with a red box. Below this, there are input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. There are also radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section has dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'.

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

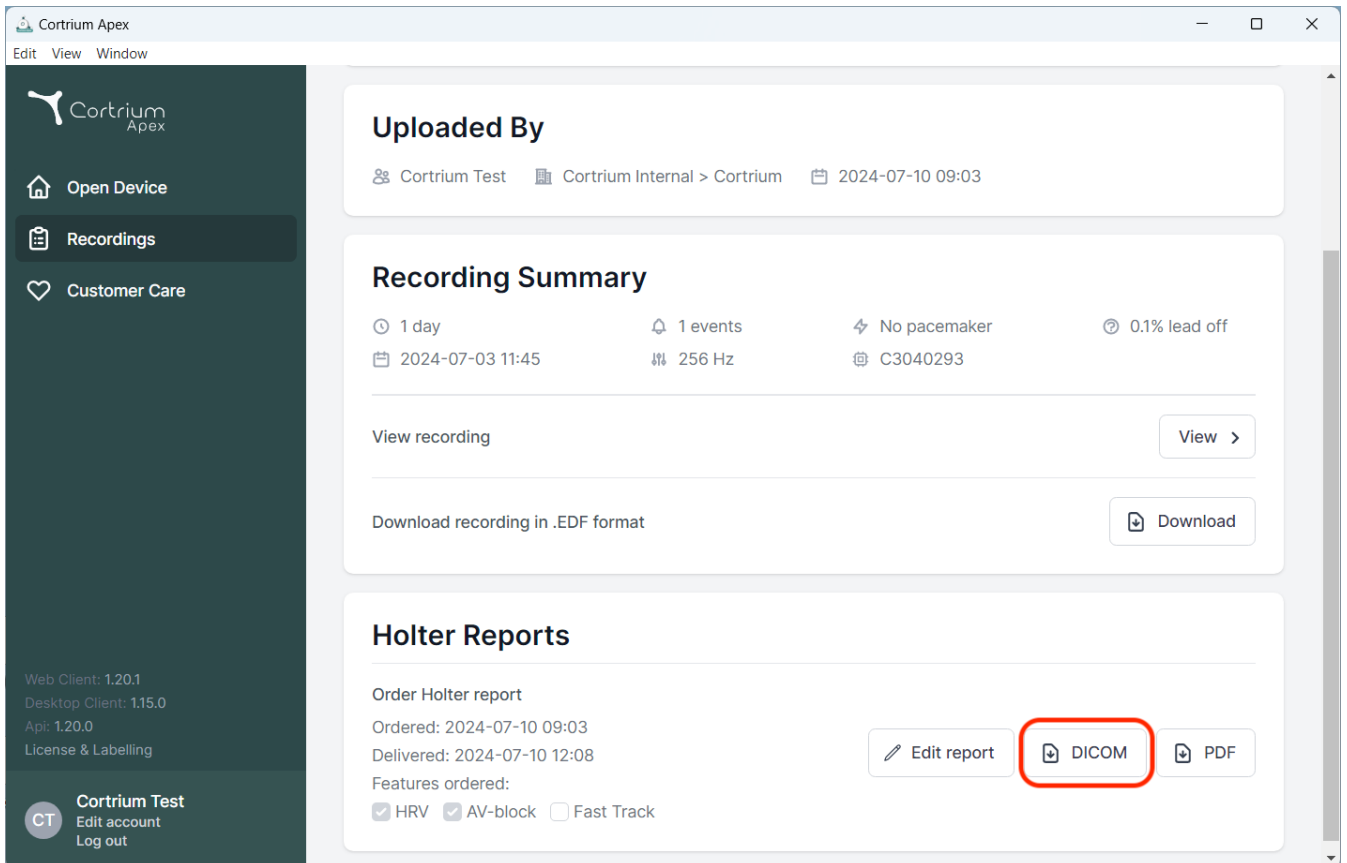
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area has a header 'Cortrium Apex' and a menu 'Edit View Window'. Below this, there are three main sections:

- Uploaded By**: Shows 'Cortrium Test', 'Cortrium Internal > Cortrium', and a timestamp '2024-07-10 09:03'.
- Recording Summary**: Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', 'C3040293', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports**: Contains an 'Order Holter report' section with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

At the bottom left of the sidebar, there is a 'CT' icon and the text 'Cortrium Test', 'Edit account', and 'Log out'. Version information is also present: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'.

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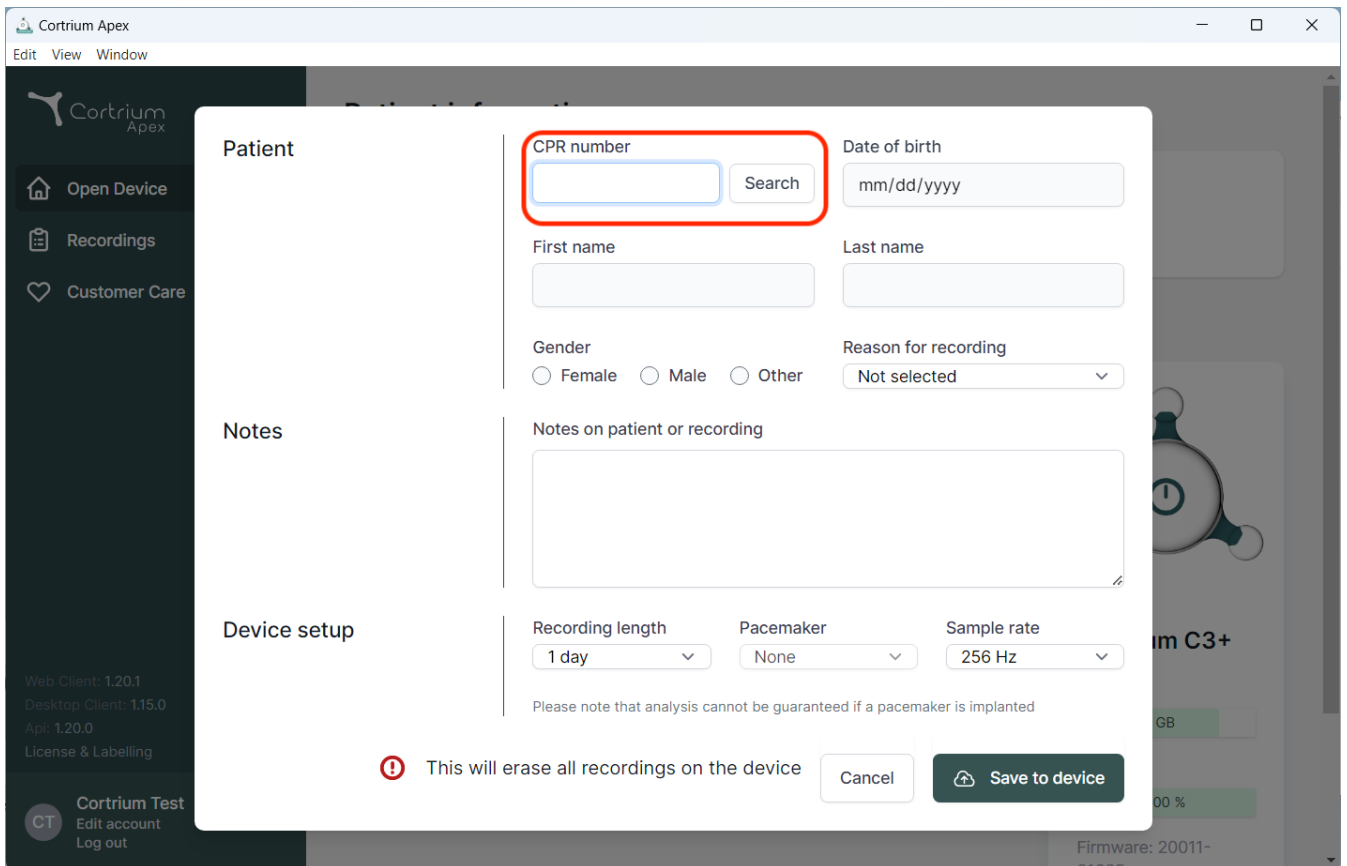The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying various fields for patient information. The "CPR number" field is highlighted with a red box, and a "Search" button is next to it. Other fields include "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), and "Notes on patient or recording". The "Device setup" section includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". Buttons for "Cancel" and "Save to device" are also visible.

Patient

CPR number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length Pacemaker Sample rate

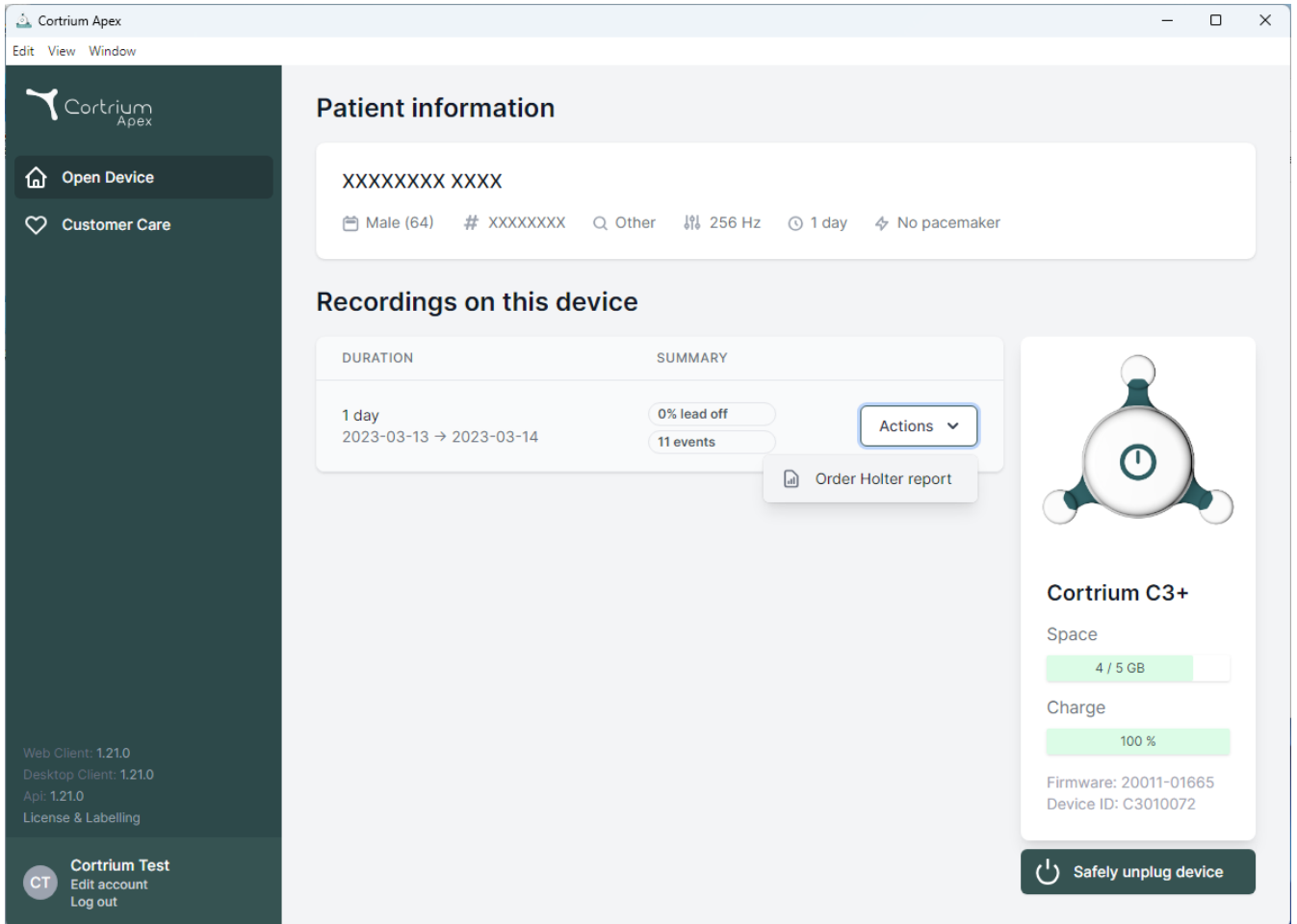
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with columns for 'DURATION' and 'SUMMARY'. The table contains one entry for '1 day' (2023-03-13 → 2023-03-14) with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right side, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB, 'Charge' at 100%, and firmware/device ID information. A 'Safely unplug device' button is located at the bottom of this card.

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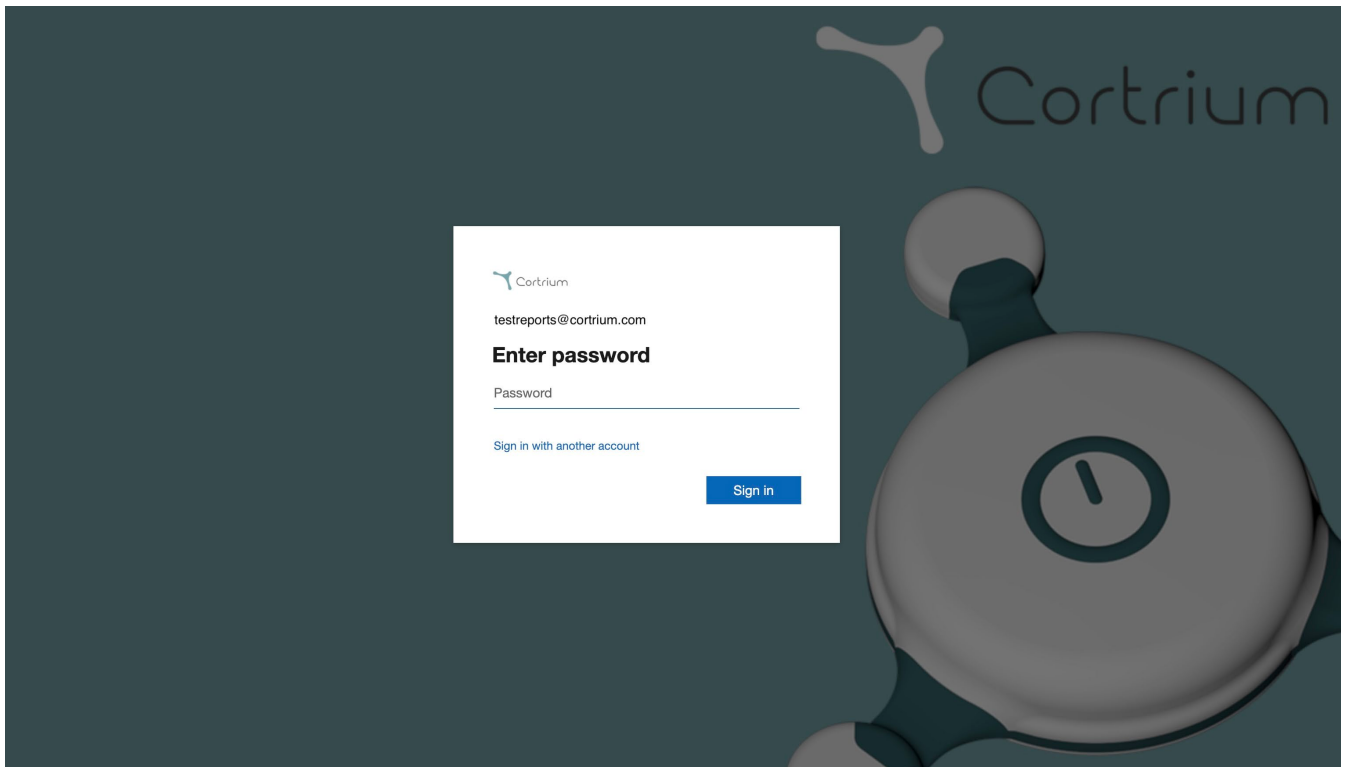
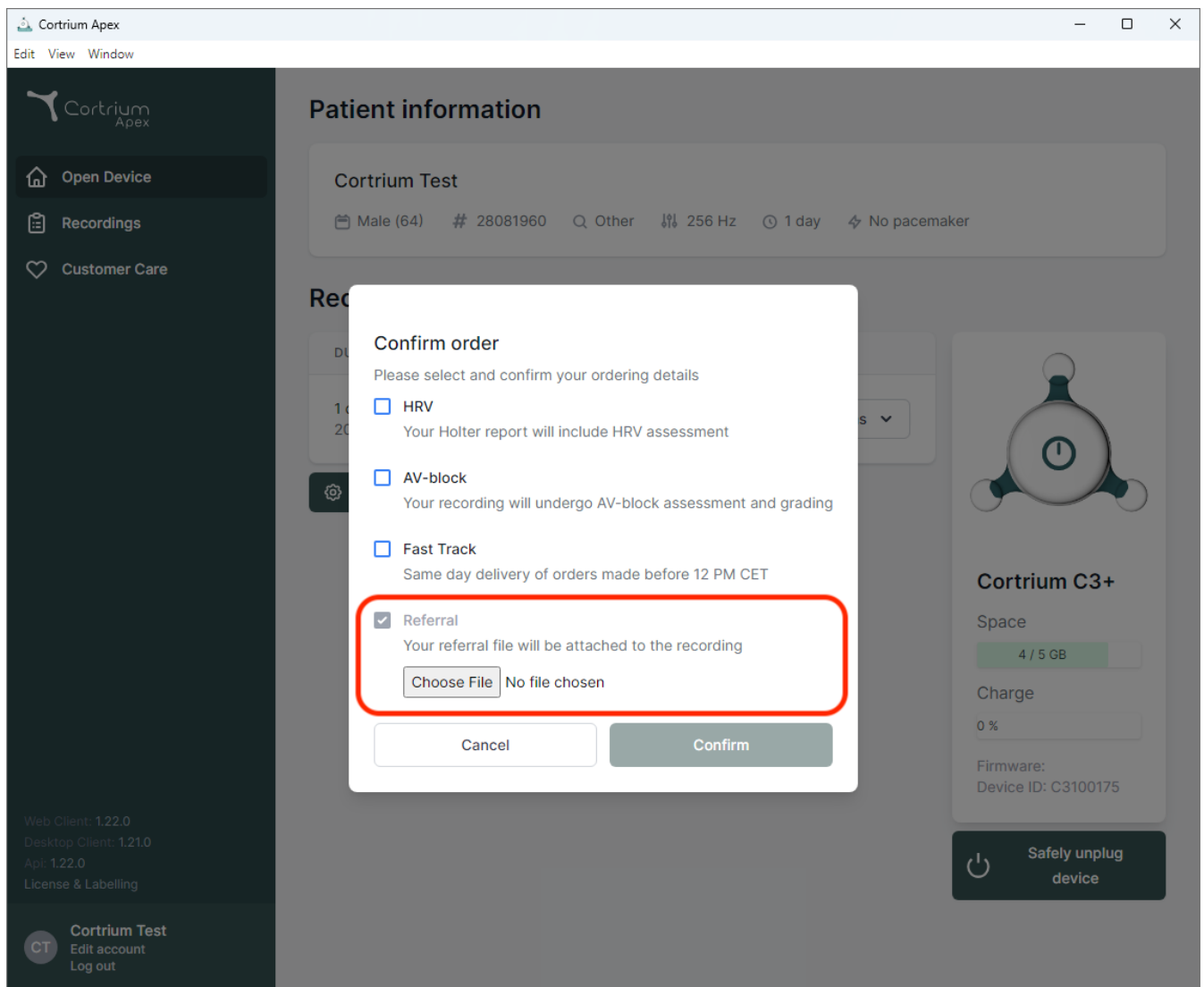
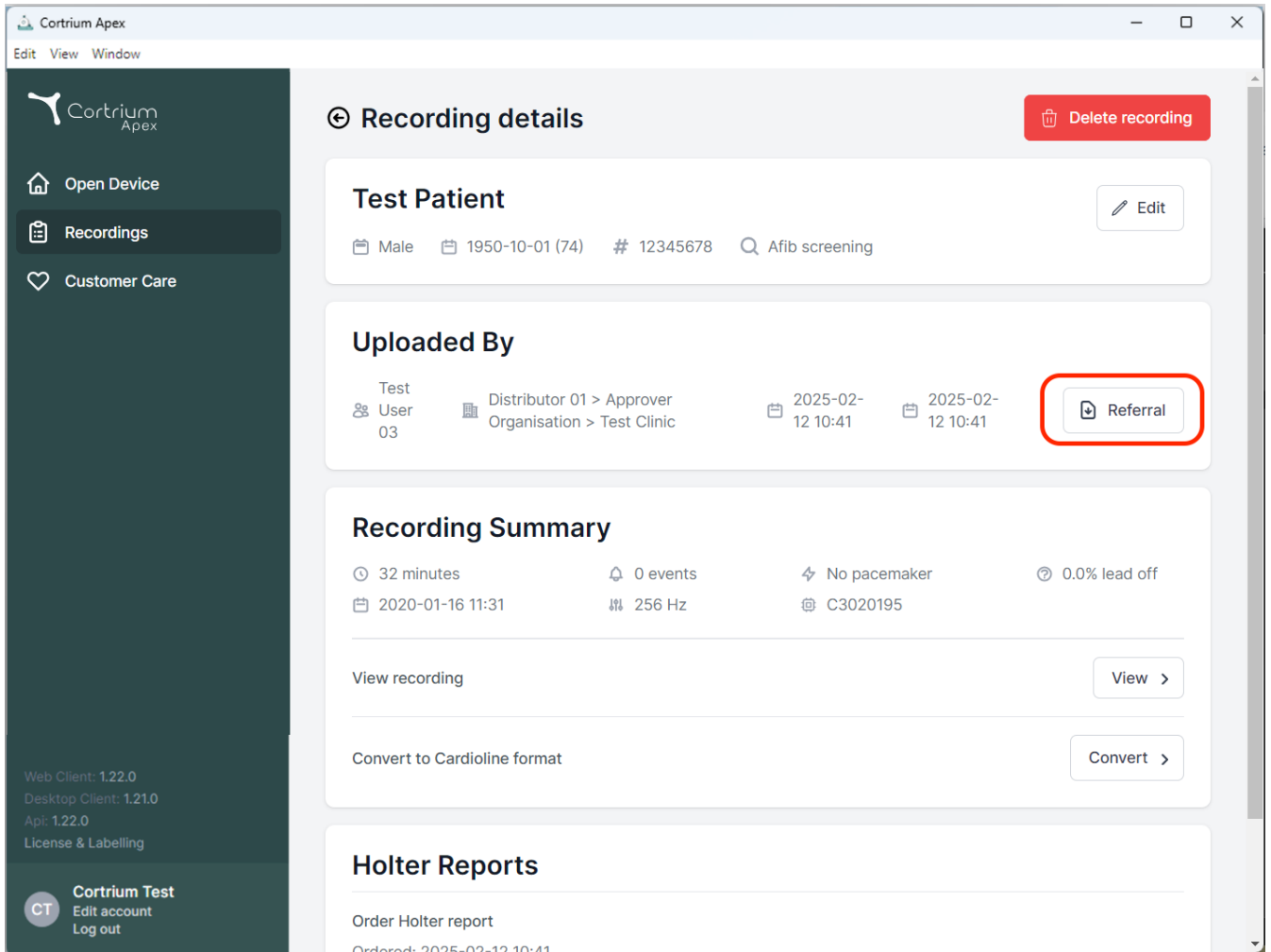
The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right corner. Below this, the 'Test Patient' section shows details for a male patient born on 1950-10-01 (74 years old), with ID # 12345678 and a note 'Afib screening'. An 'Edit' button is available for this section. The 'Uploaded By' section lists 'Test User 03' as the uploader, with the recording approved by 'Distributor 01 > Approver' and 'Organisation > Test Clinic'. It also shows the upload date as 2025-02-12 10:41. A 'Referral' button is highlighted with a red rectangle. The 'Recording Summary' section provides key metrics: 32 minutes duration, 0 events, no pacemaker, and 0.0% lead off. It also lists the recording date as 2020-01-16 11:31 and a frequency of 256 Hz. Below the summary, there are buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section at the bottom includes an 'Order Holter report' button and a note 'Ordered: 2025-02-12 10:41'.

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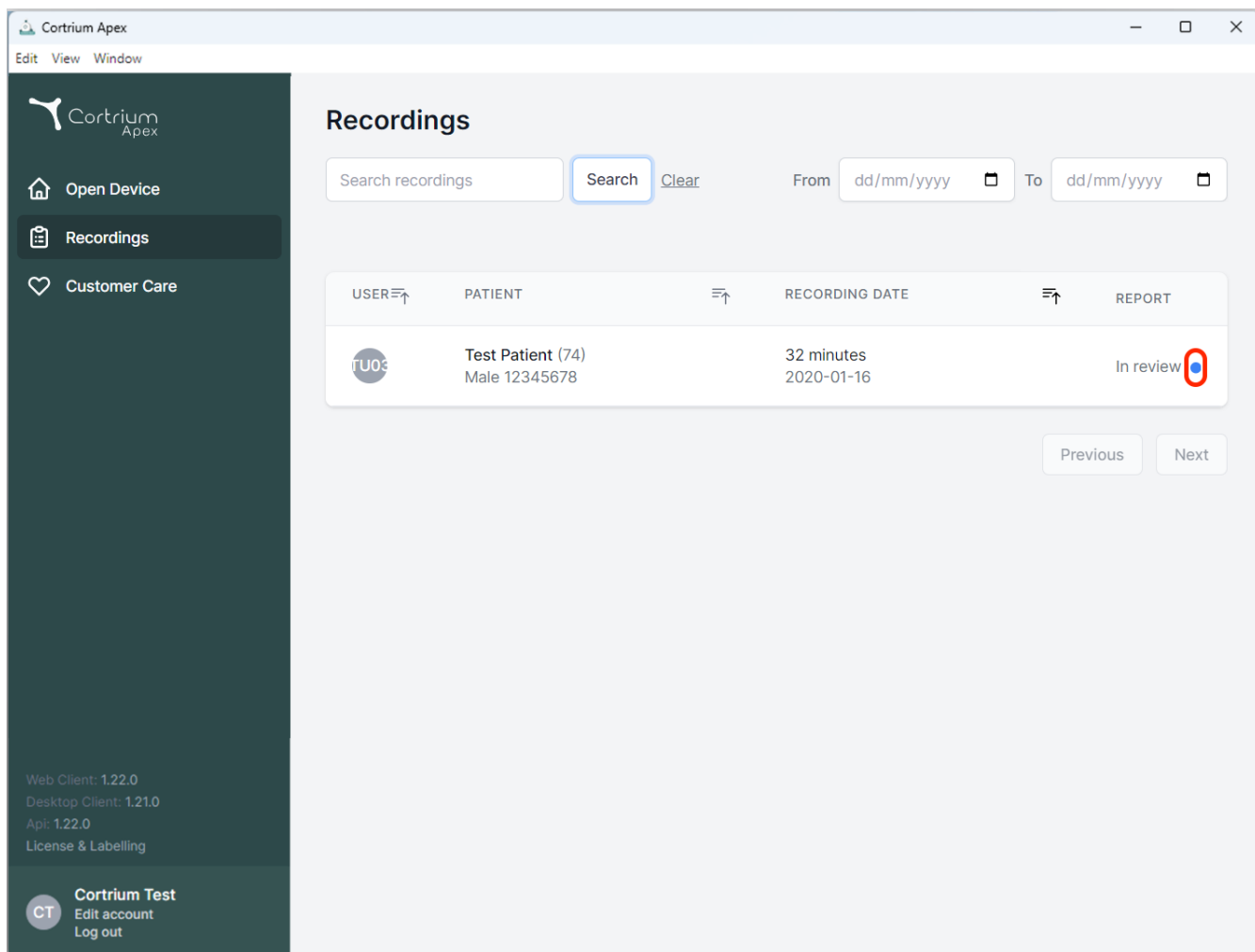Recordings

Search recordings **Search** [Clear](#) From dd/mm/yyyy To dd/mm/yyyy

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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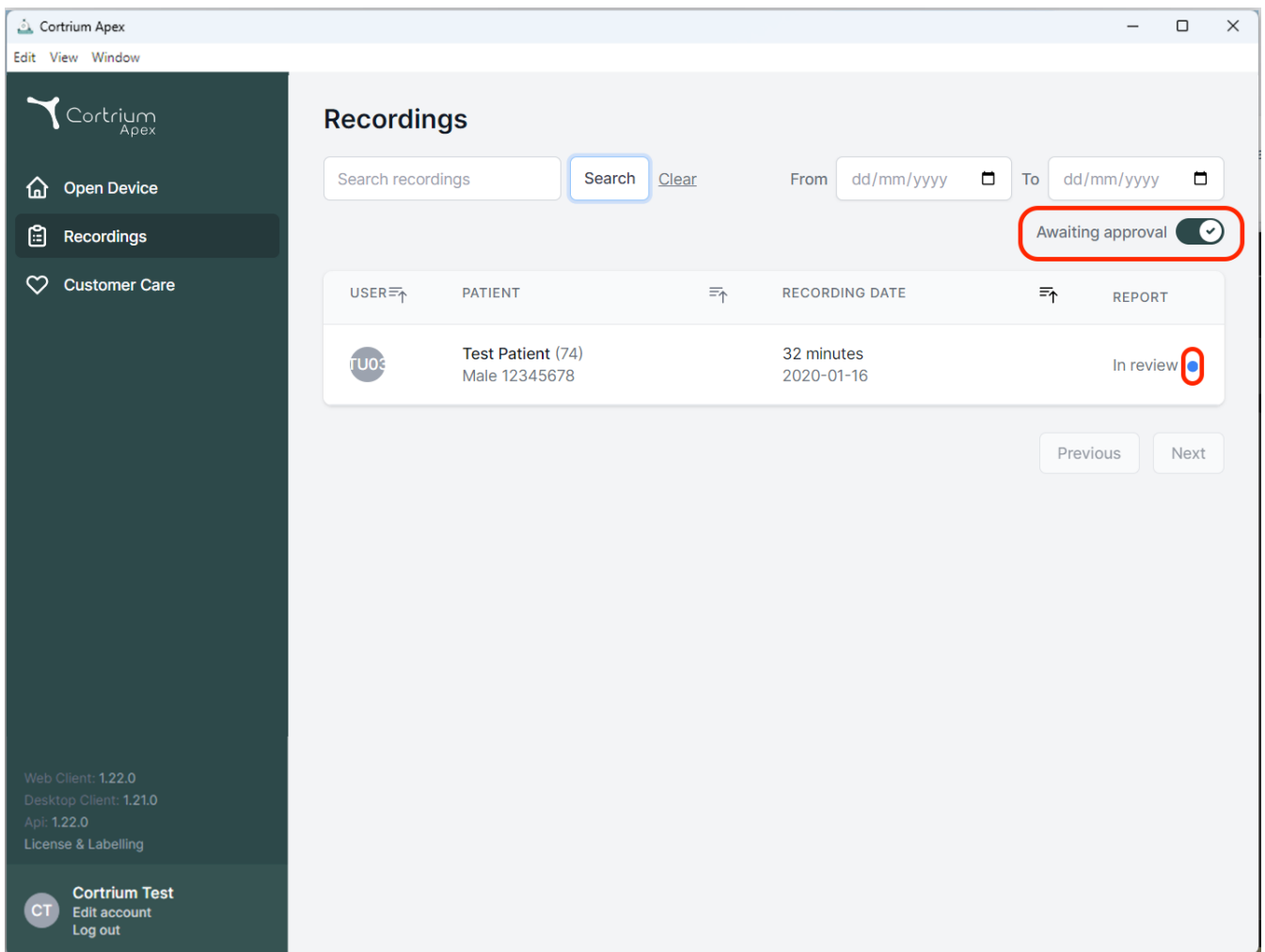
Recordings

Search recordings From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●

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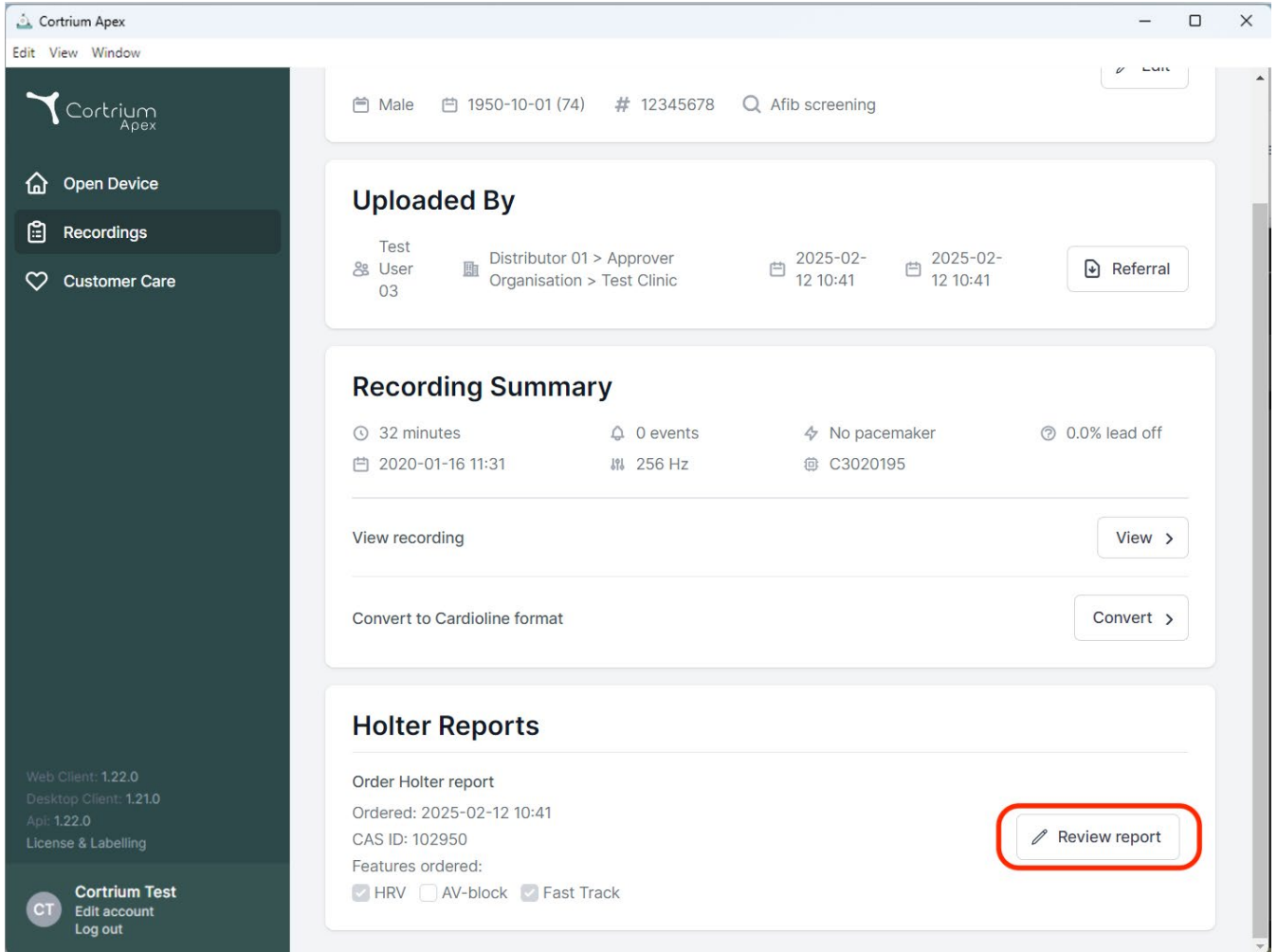The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings (highlighted), and Customer Care. The main content area shows patient information (Male, 1950-10-01, # 12345678, Afib screening) and a section titled 'Uploaded By' with details about the test user and distributor. Below this is a 'Recording Summary' section showing recording duration (32 minutes), events (0), pacemaker status (No pacemaker), and lead off percentage (0.0%). It also includes a 'View recording' button and a 'Convert to Cardioline format' button. The 'Holter Reports' section shows an 'Order Holter report' with details like 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A red box highlights the 'Review report' button in the Holter Reports section.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

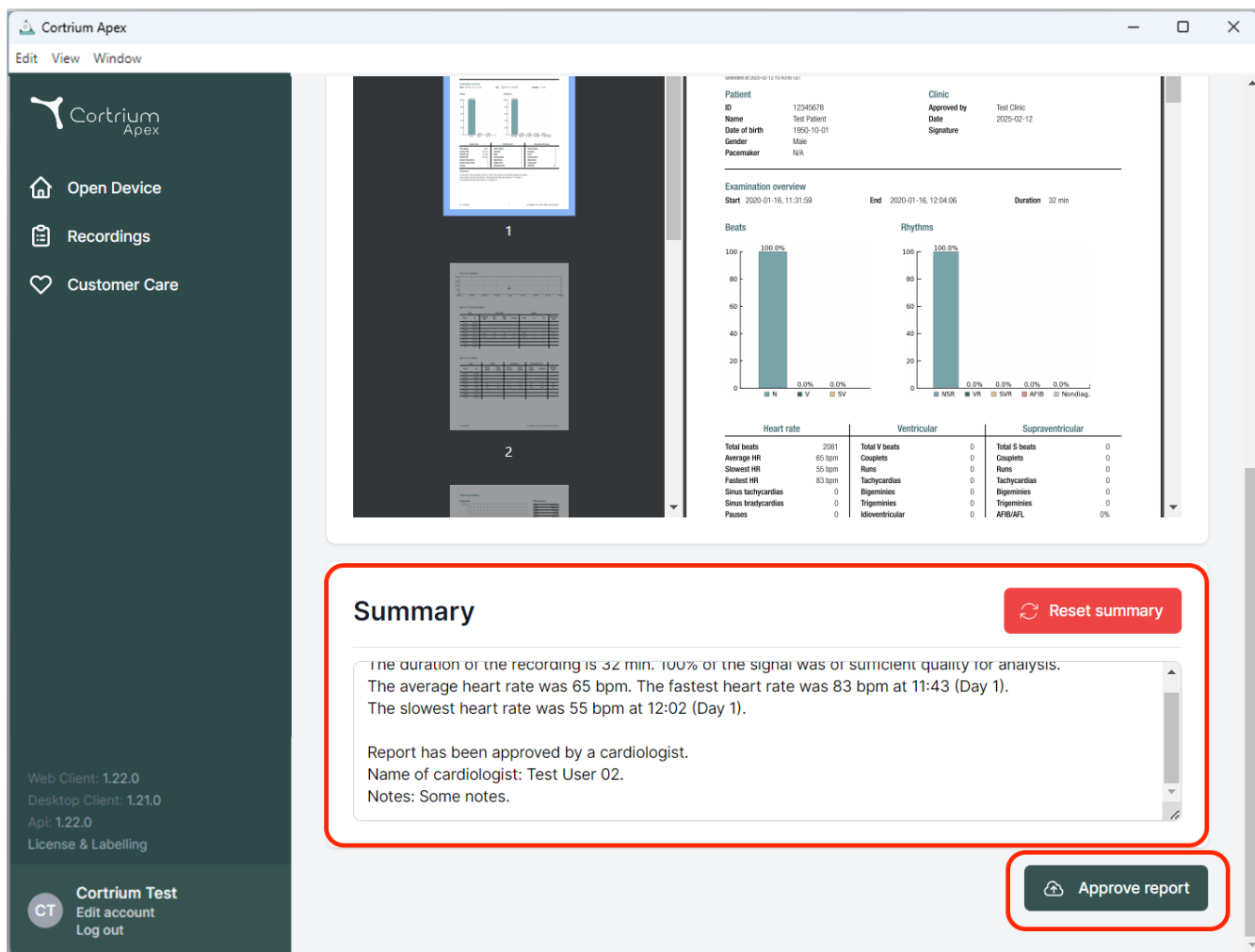
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary [Reset summary](#)

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

[Approve report](#)

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark
E-post: info@cortrium.com • Nettsted: www.cortrium.com

Cortrium Apex UDI(01)05745000379033