Cortrium Apex 1.22

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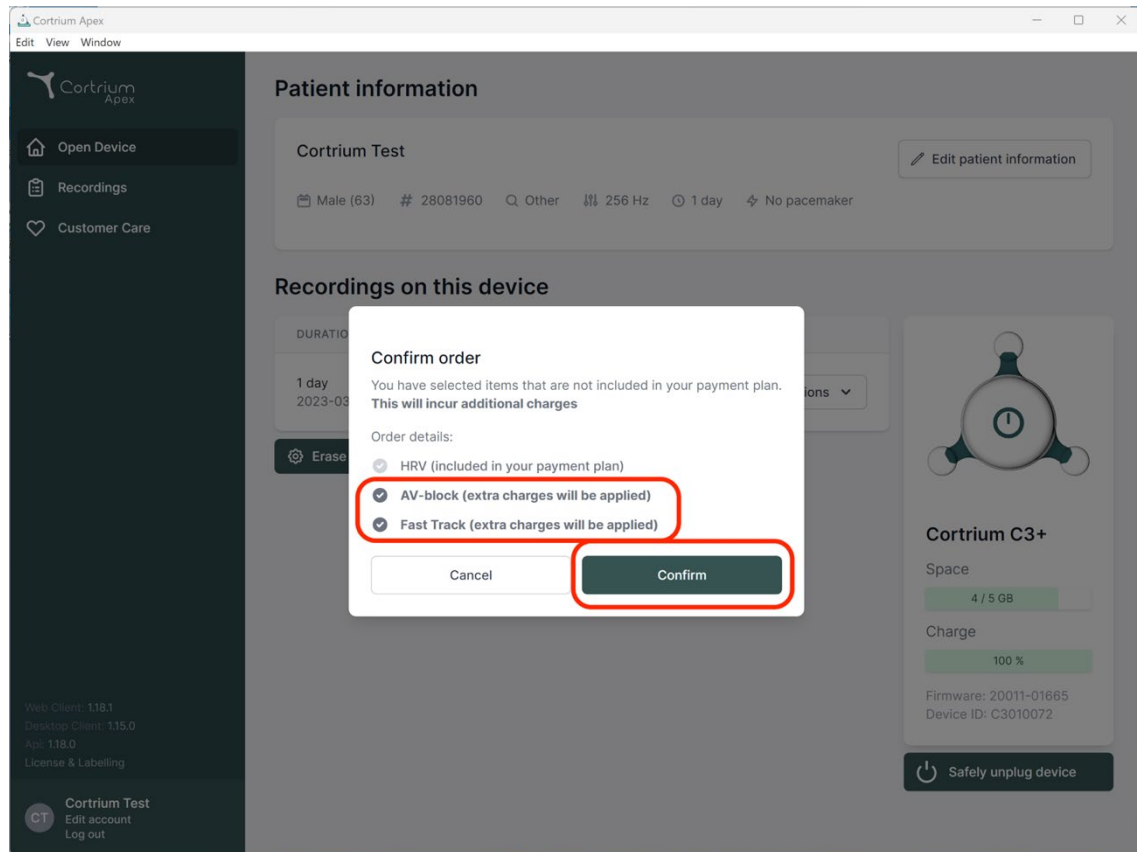
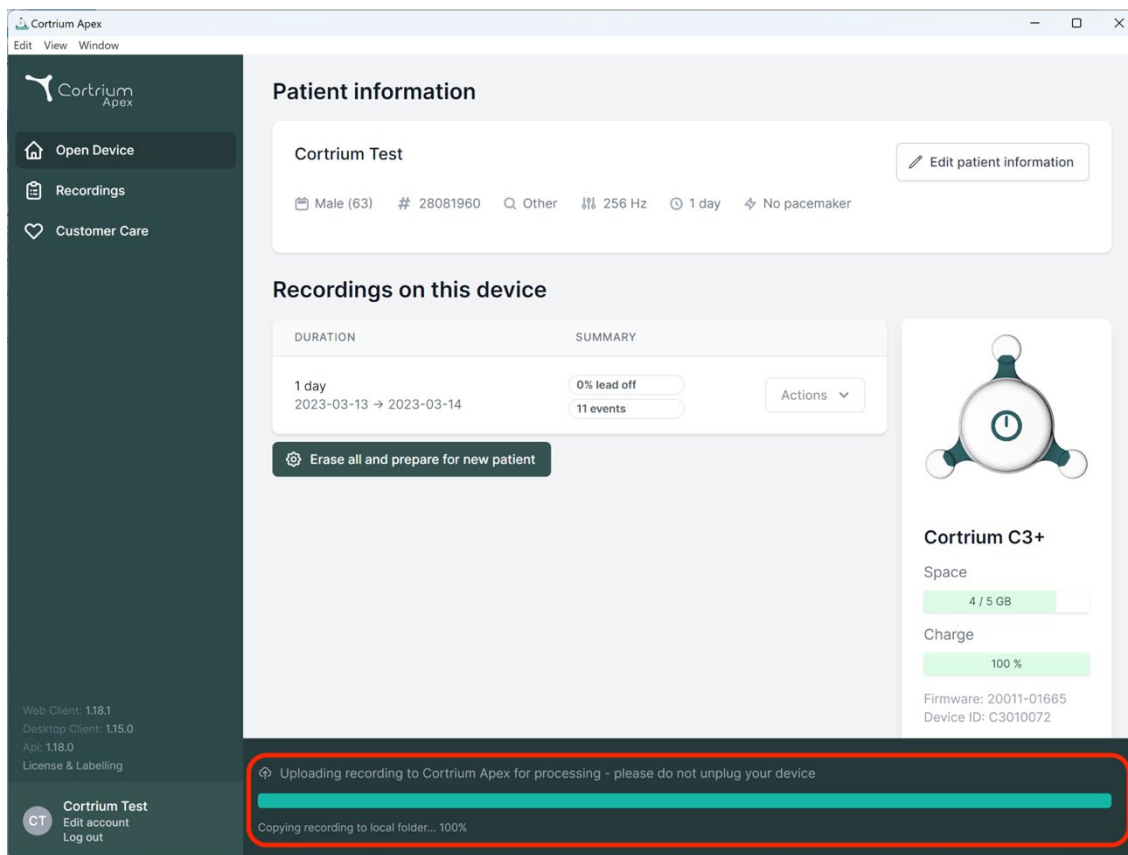
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including patient demographics and device settings. Below this is a 'Recordings on this device' section with a table showing a 1-day recording period from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom of the interface, which contains the text: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. On the right side of the interface, there is a device status card for 'Cortrium C3+' showing space usage (4 / 5 GB), charge level (100%), and firmware/device ID information.

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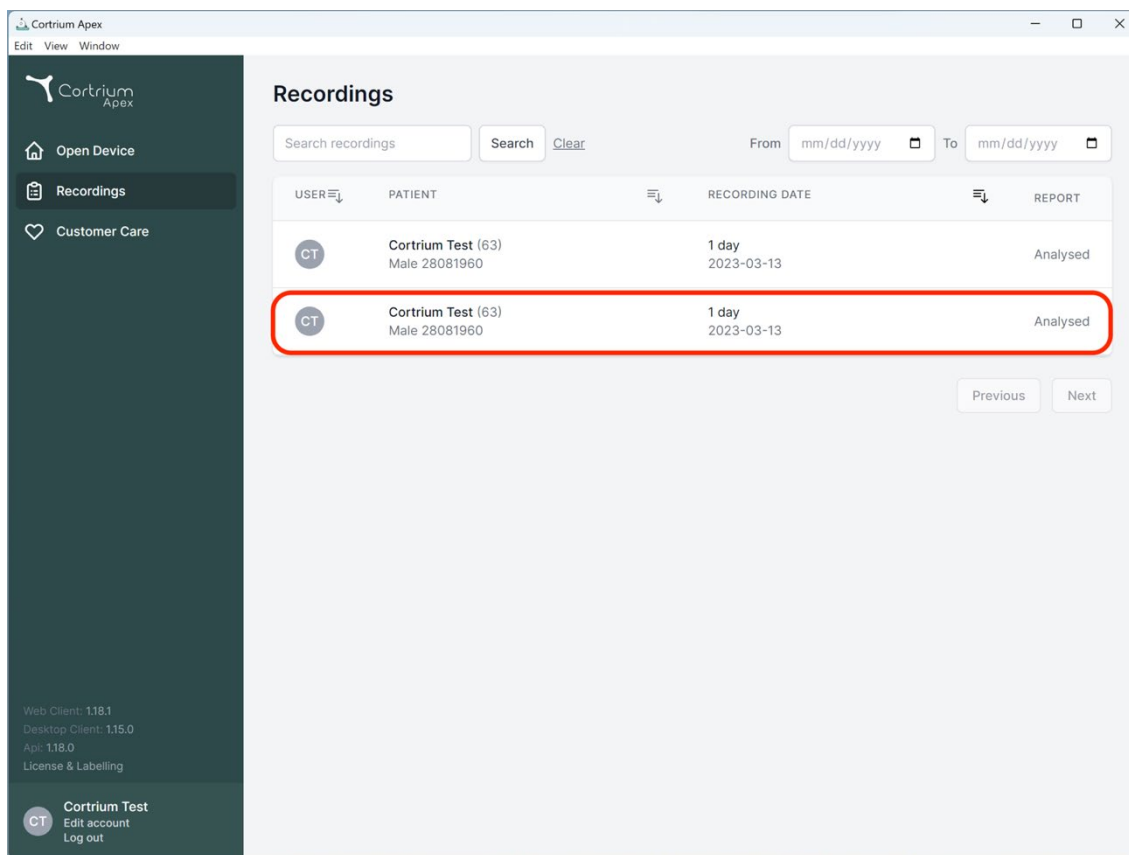
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. At the bottom of the sidebar, there is a 'Cortrium Test' button with a 'CT' icon and links for 'Edit account' and 'Log out'. The main area shows a table of recordings. The table has four columns: 'USER' (with a dropdown arrow), 'PATIENT' (with a dropdown arrow), 'RECORDING DATE' (with a dropdown arrow), and 'REPORT'. There are two rows of data. The first row shows 'Cortrium Test (63)' for patient 'Male 28081960' with a '1 day' recording date from '2023-03-13', and the report status is 'Analysed'. The second row is identical but is highlighted with a red border. Above the table, there is a search bar labeled 'Search recordings' with 'Search' and 'Clear' buttons, and date filters 'From' and 'To' in 'mm/dd/yyyy' format. Below the table, there are 'Previous' and 'Next' navigation buttons. The top of the window shows the 'Cortrium Apex' title bar with 'Edit', 'View', and 'Window' menus.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT

Cortrium Test

Edit account
Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
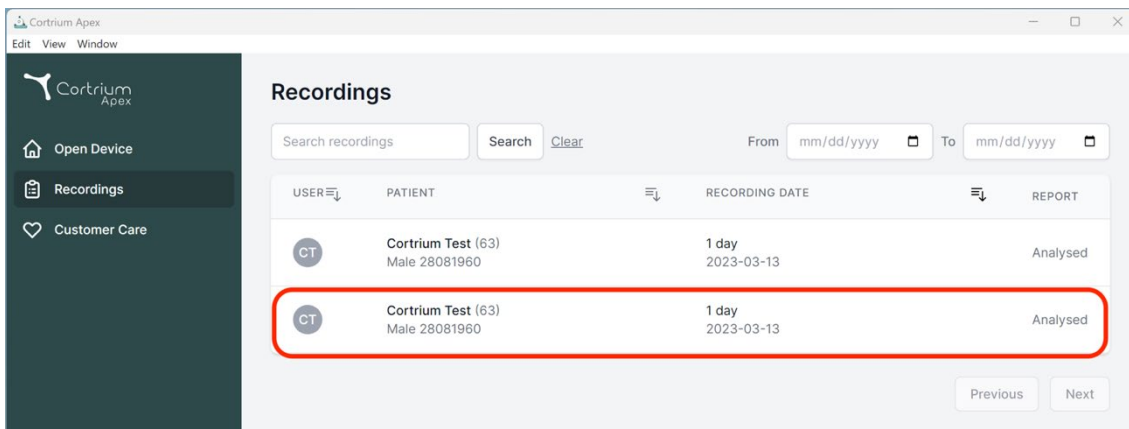
Recordings

Search recordings Search [Clear](#) From To

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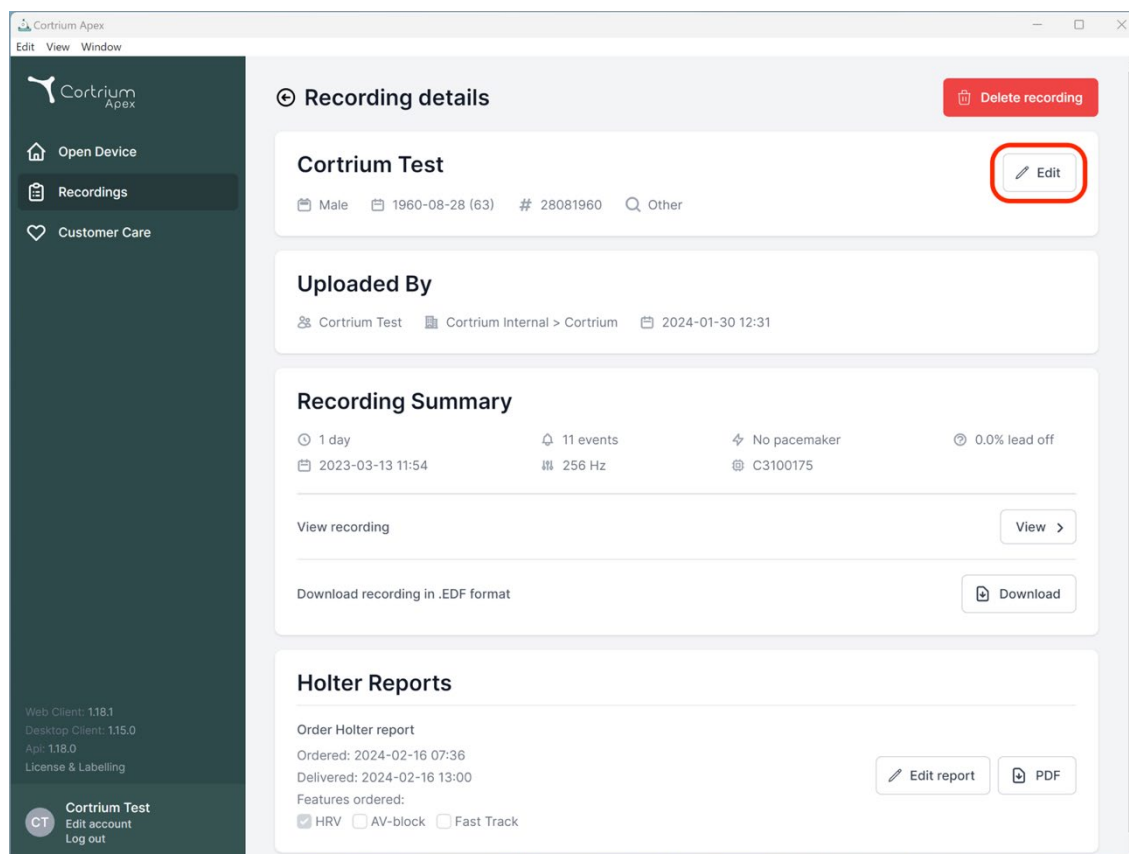Recording details [Delete recording](#)

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View recording [View >](#)

Download recording in .EDF format [Download](#)

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
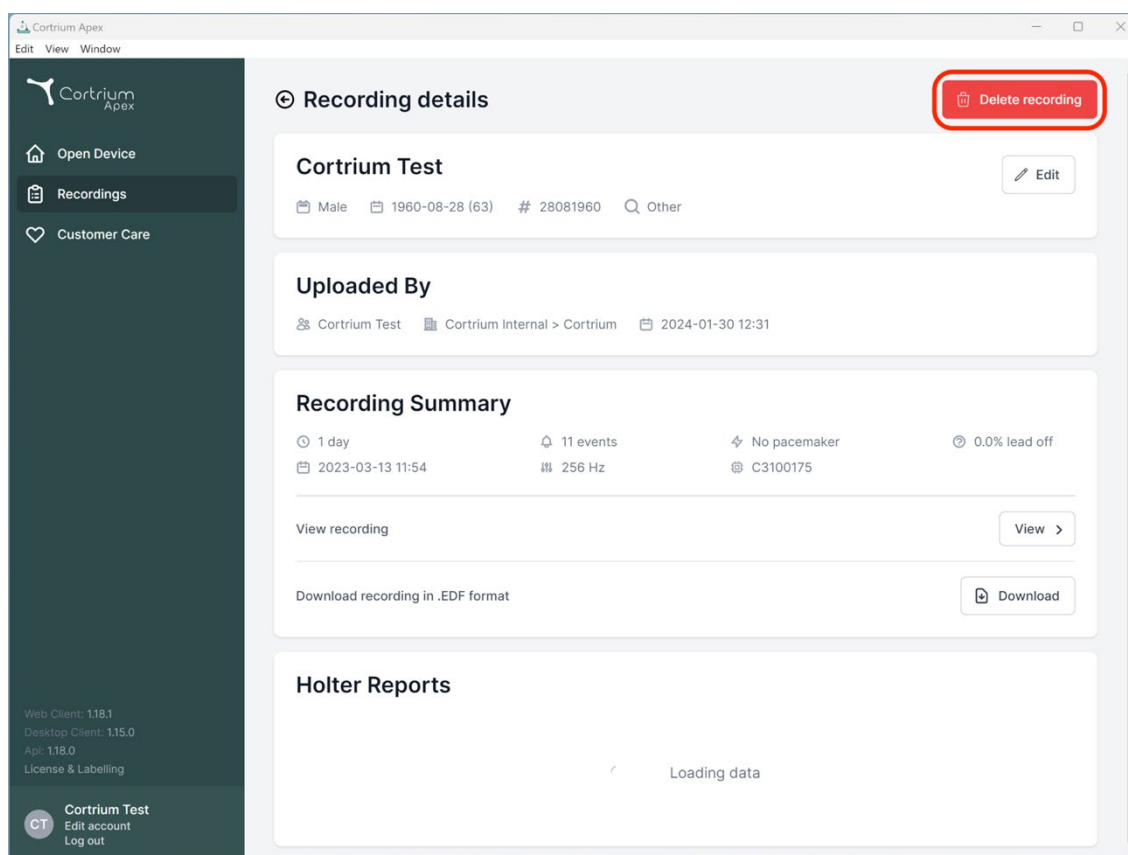
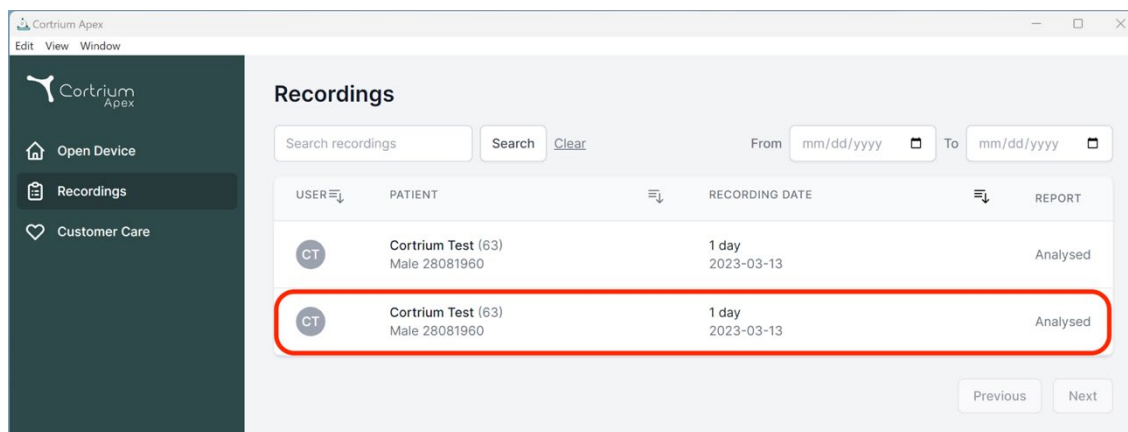
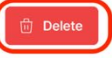
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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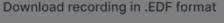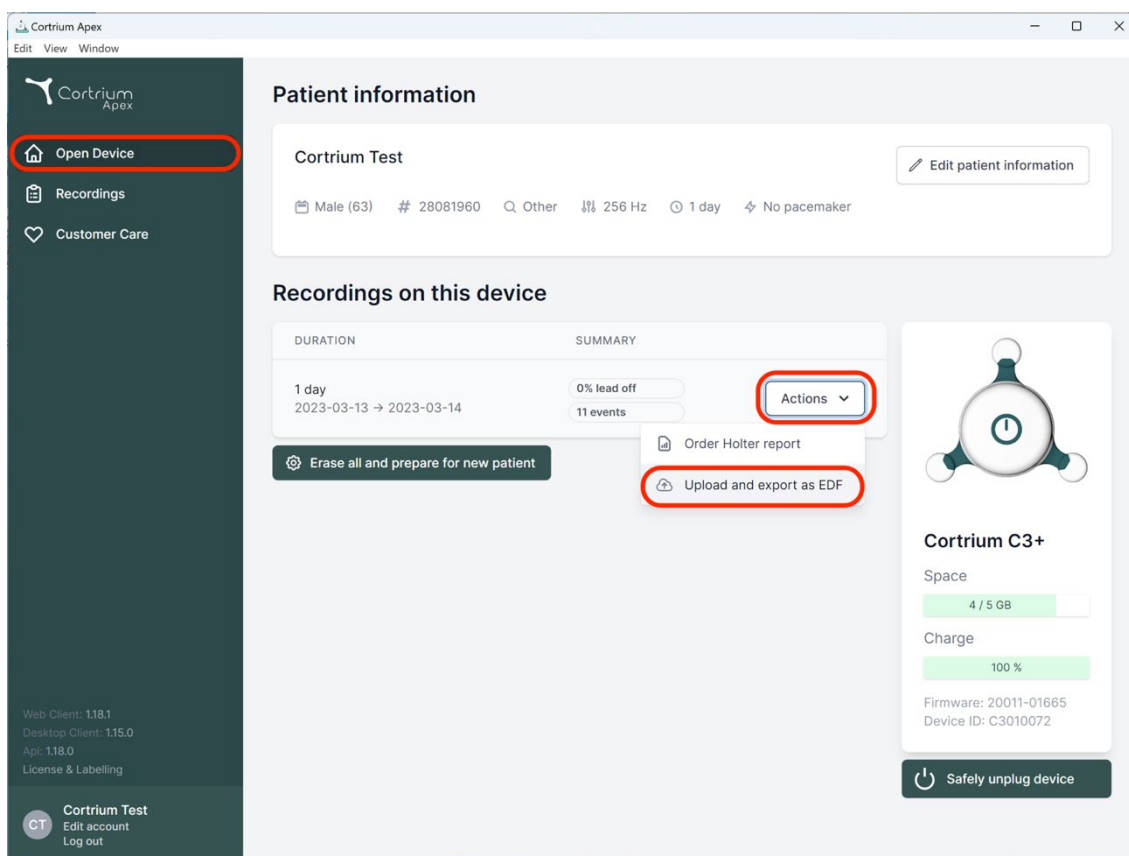
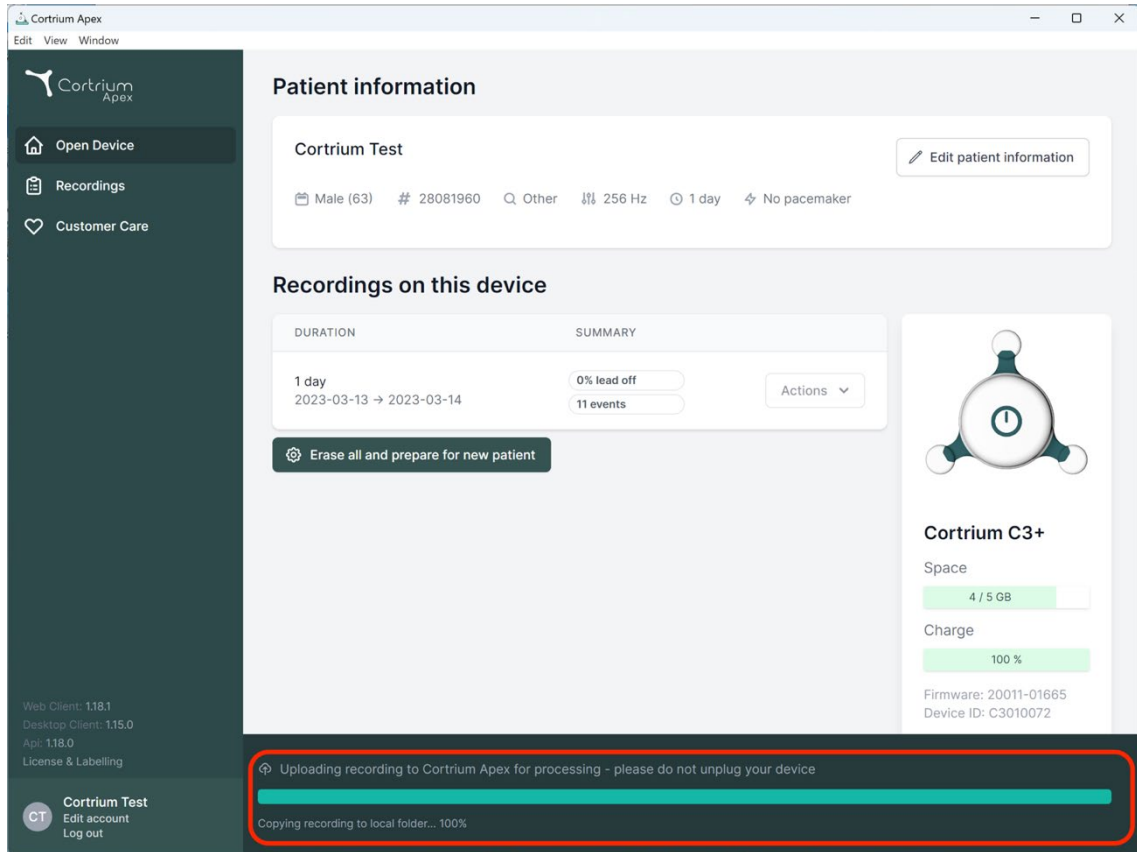
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

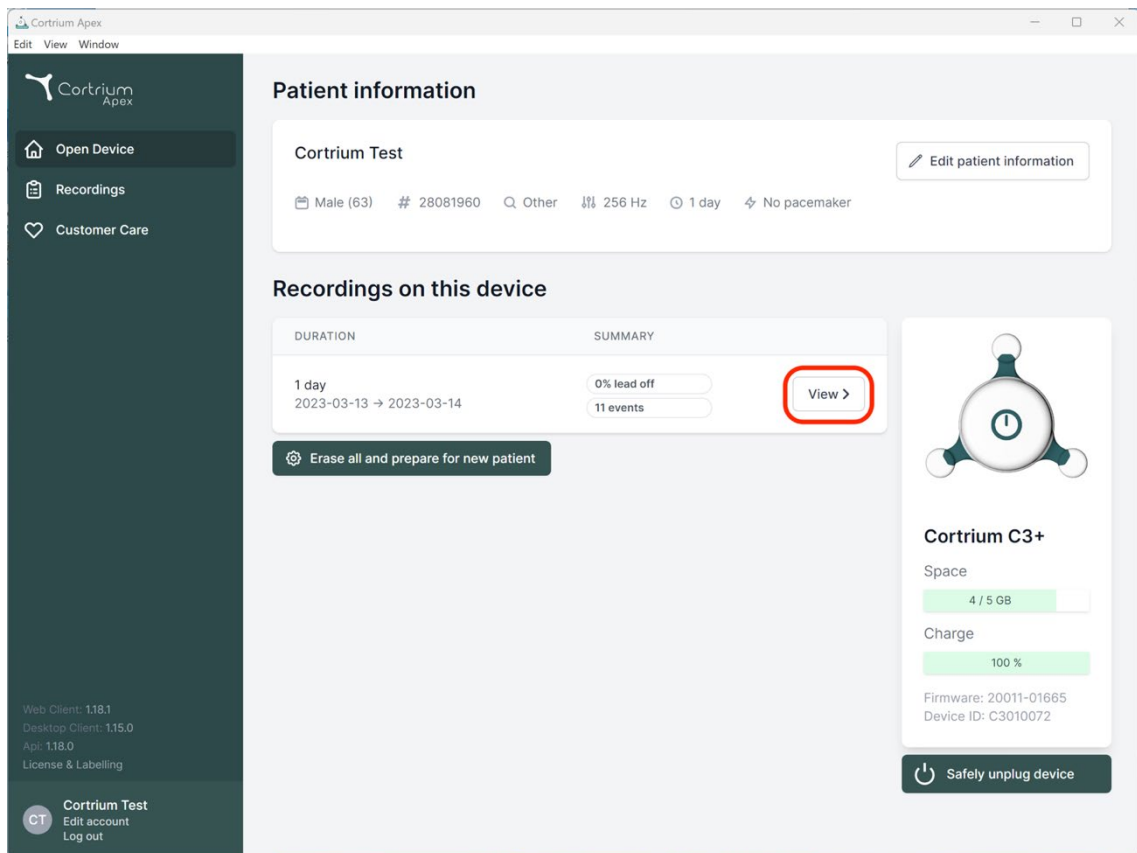
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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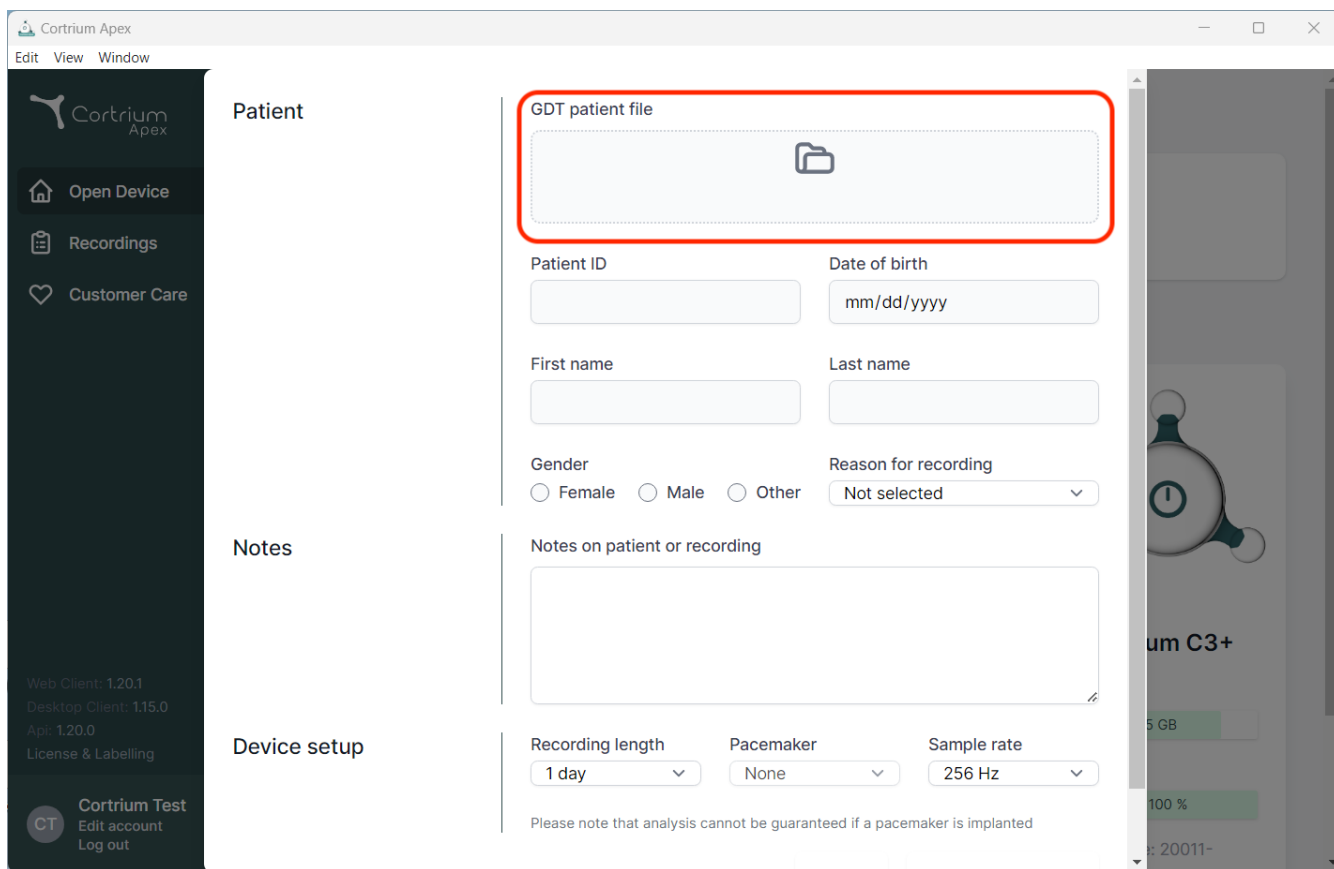
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

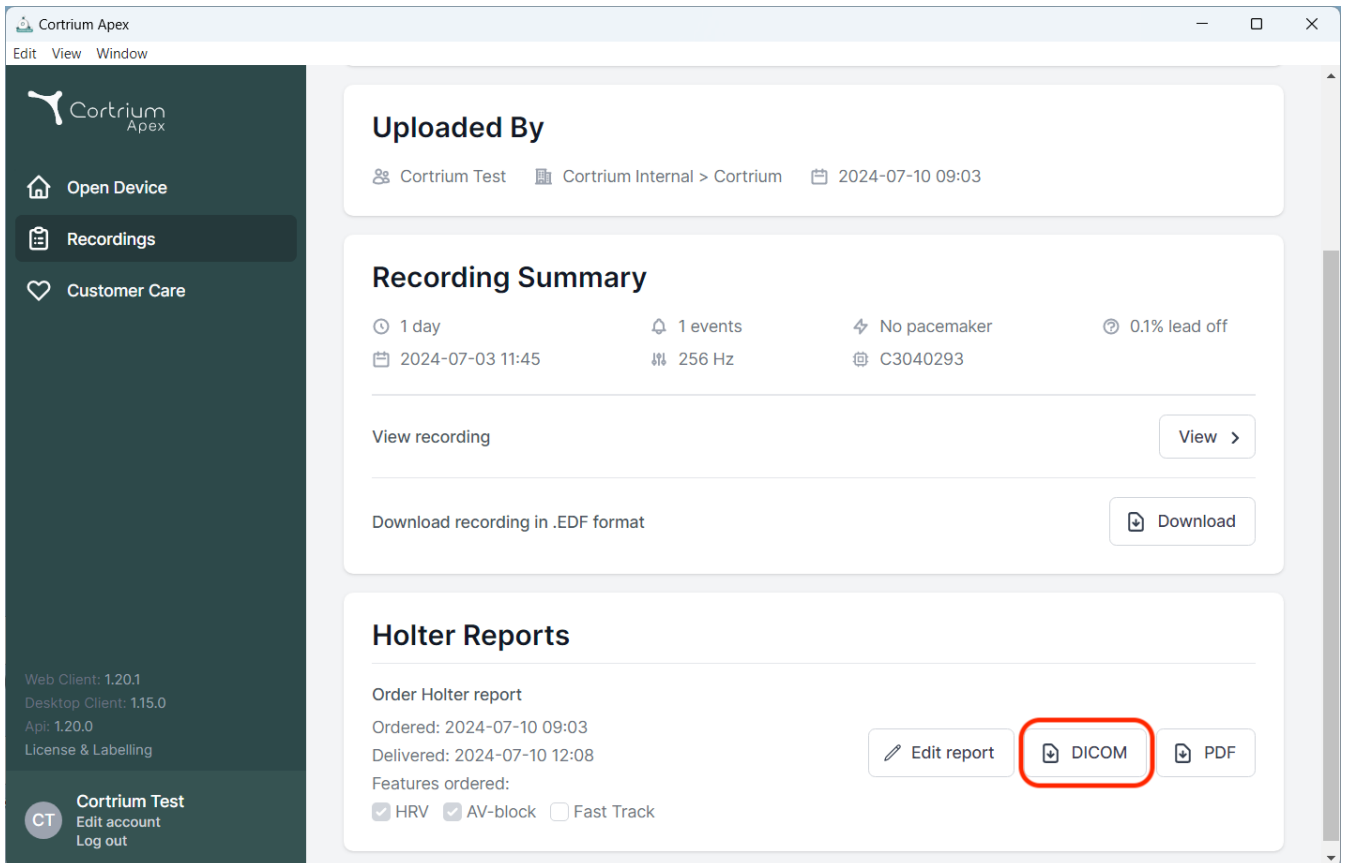
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section features a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a heart icon and the text "um C3+", "5 GB", "100 %", and "20011-".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying recording details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'. Below these buttons, the 'Features ordered' section shows checkboxes for 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked).

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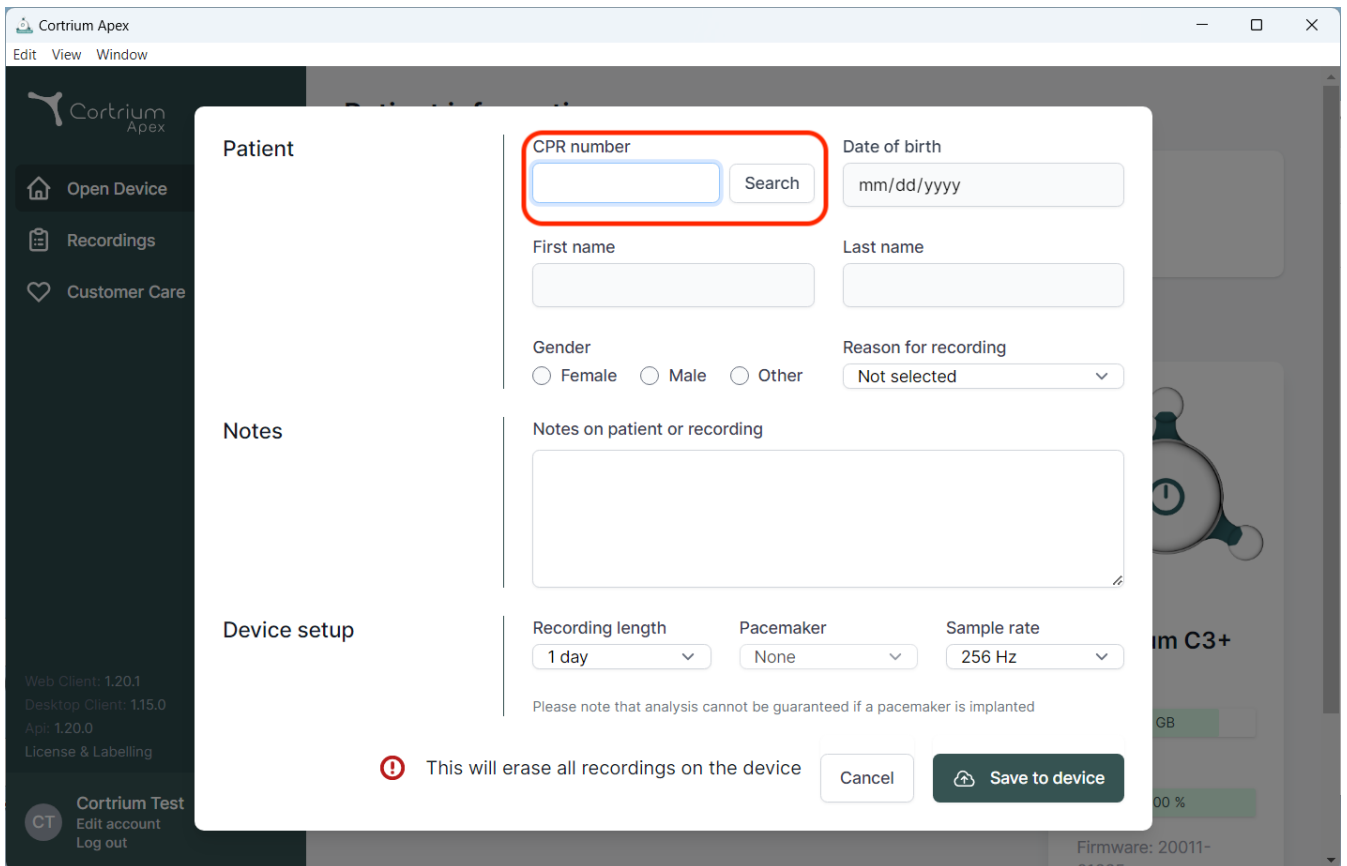Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient

CPR number Search

Date of birth

First name

Last name

Gender
☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

Pacemaker

Sample rate

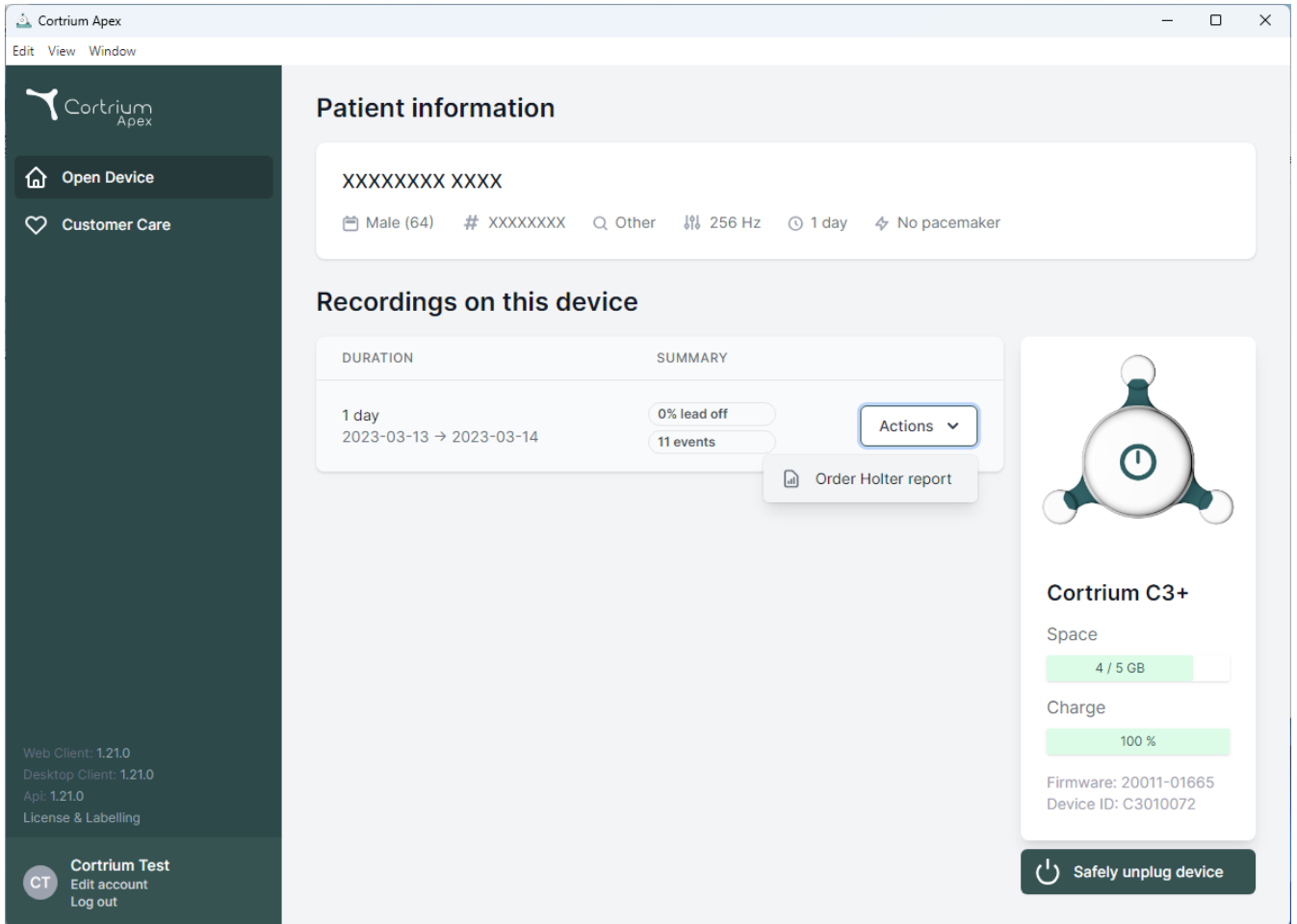
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), 'Customer Care' (with a heart icon), and a footer section with 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', 'License & Labelling', and a 'Cortrium Test' button with 'Edit account' and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section displays a table with columns 'DURATION' and 'SUMMARY'. The first row shows a recording for '1 day' from '2023-03-13' to '2023-03-14', with '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right side, a device status card for 'Cortrium C3+' shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of the card.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

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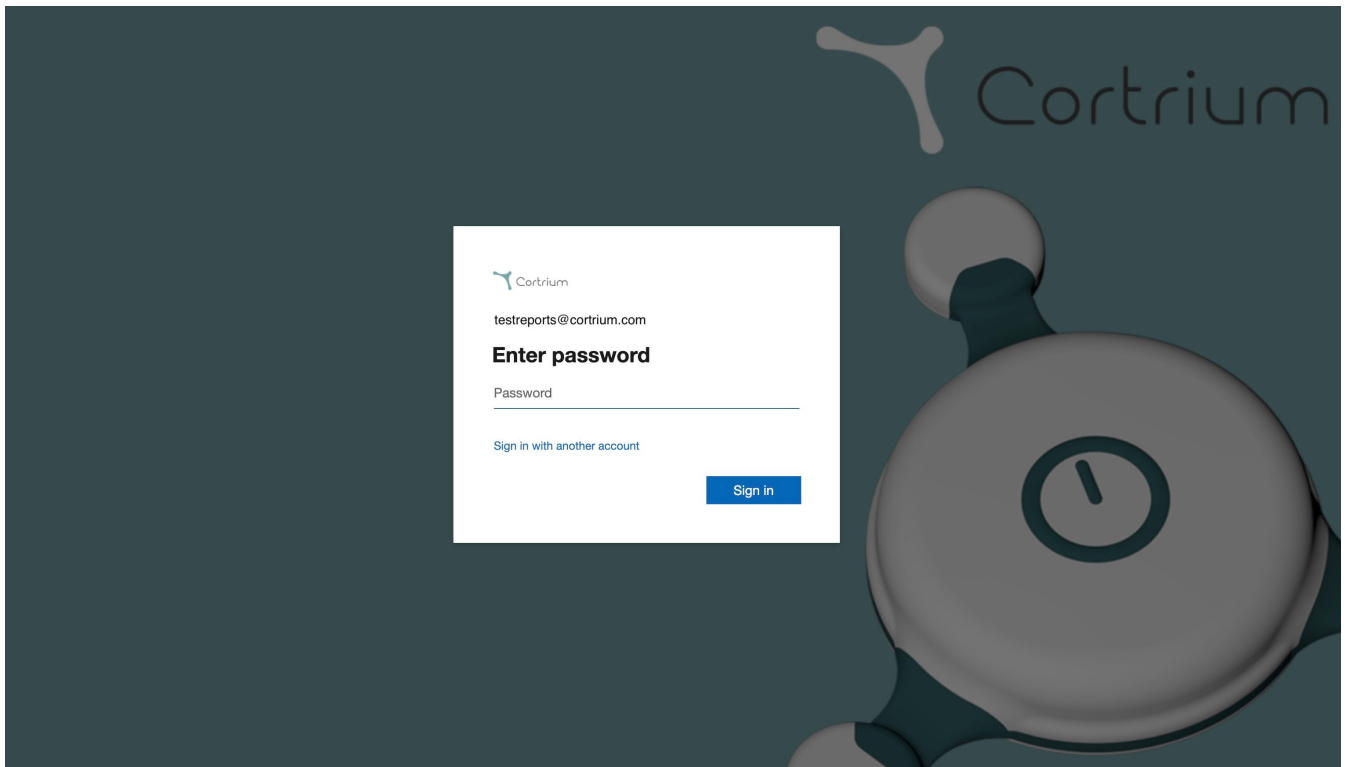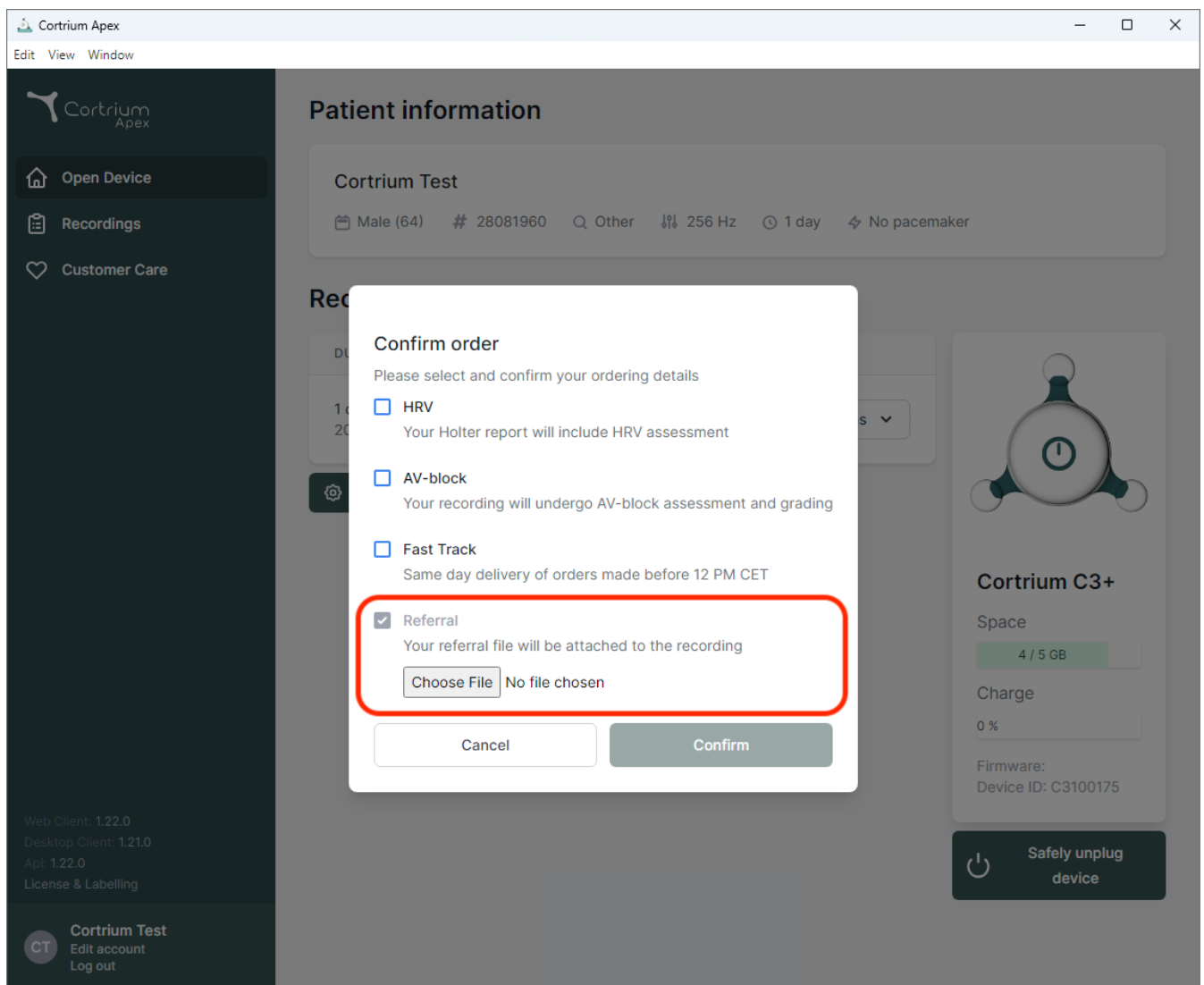
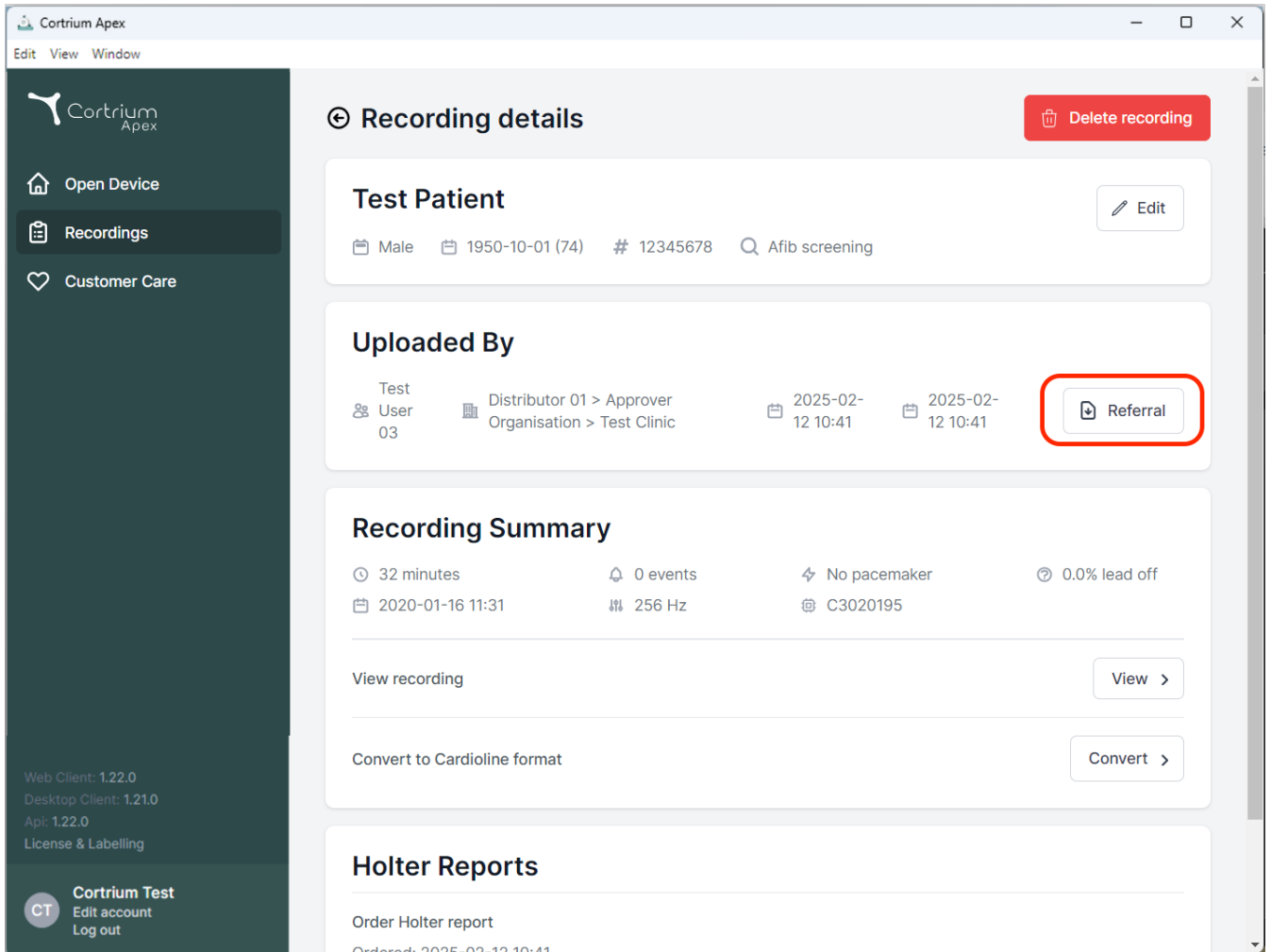
The screenshot displays the 'Recording details' page in the Cortrium Apex application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is titled 'Recording details' and features a 'Delete recording' button in the top right corner. Below this, the 'Test Patient' section shows details for a male patient born on 1950-10-01 (74 years old), with ID 12345678 and a search term 'Afib screening'. An 'Edit' button is available for this section. The 'Uploaded By' section lists the test user as 'User 03', the distributor as 'Distributor 01 > Approver', and the organization as 'Organisation > Test Clinic'. It also shows upload dates of 2025-02-12 10:41. A 'Referral' button is highlighted with a red rectangle. The 'Recording Summary' section provides key metrics: 32 minutes duration, 0 events, no pacemaker, and 0.0% lead off. It also includes the recording date (2020-01-16 11:31), frequency (256 Hz), and device ID (C3020195). At the bottom of this section are 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section at the very bottom includes an 'Order Holter report' button and a note that the report was ordered on 2025-02-12 10:41. The sidebar also contains version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0) and a 'License & Labelling' link. At the bottom of the sidebar, there is a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

Cortrium Apex

Edit View Window

Recording details Delete recording

Test Patient Edit

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

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
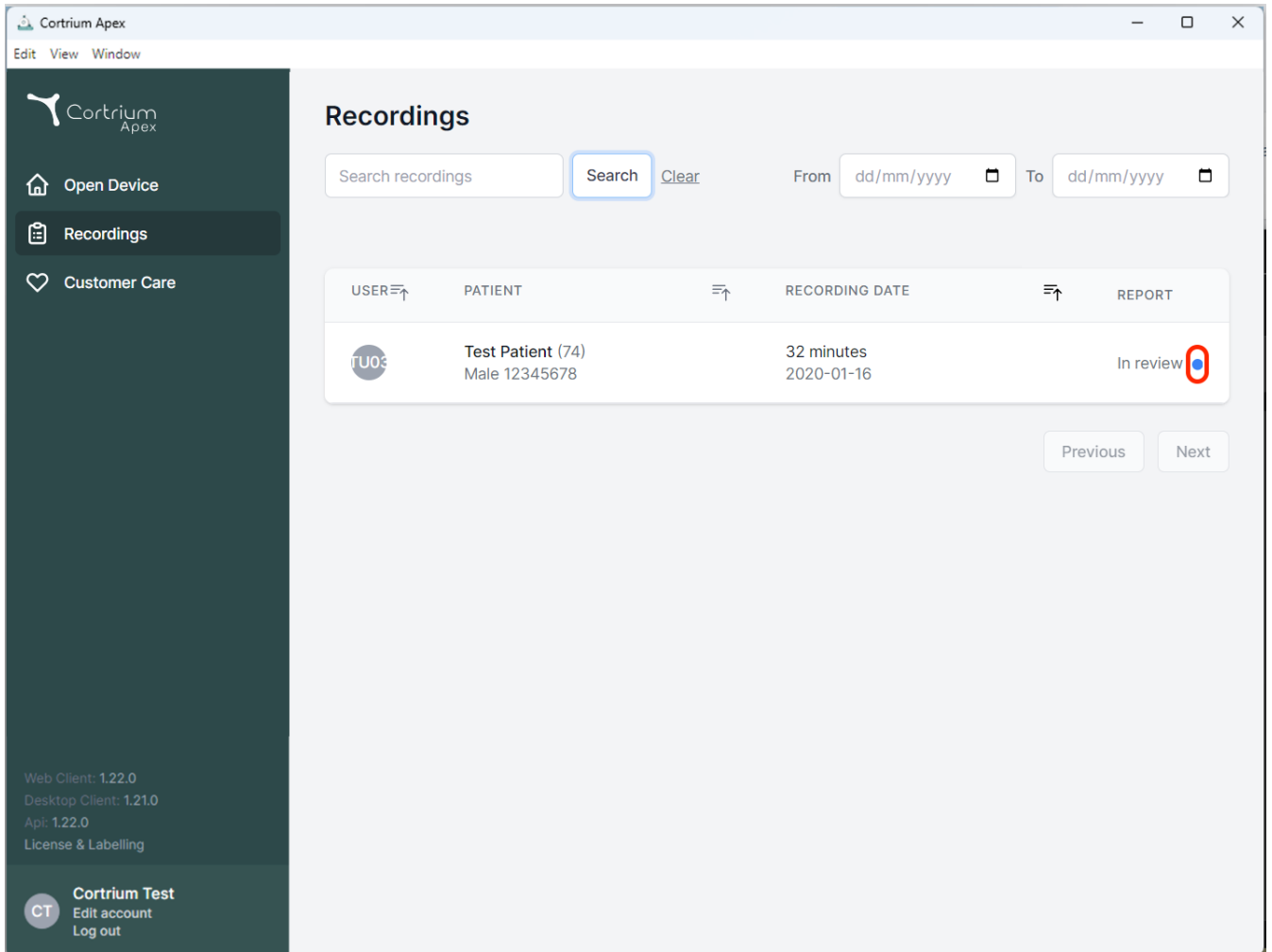
Search recordings [Clear](#)

From To

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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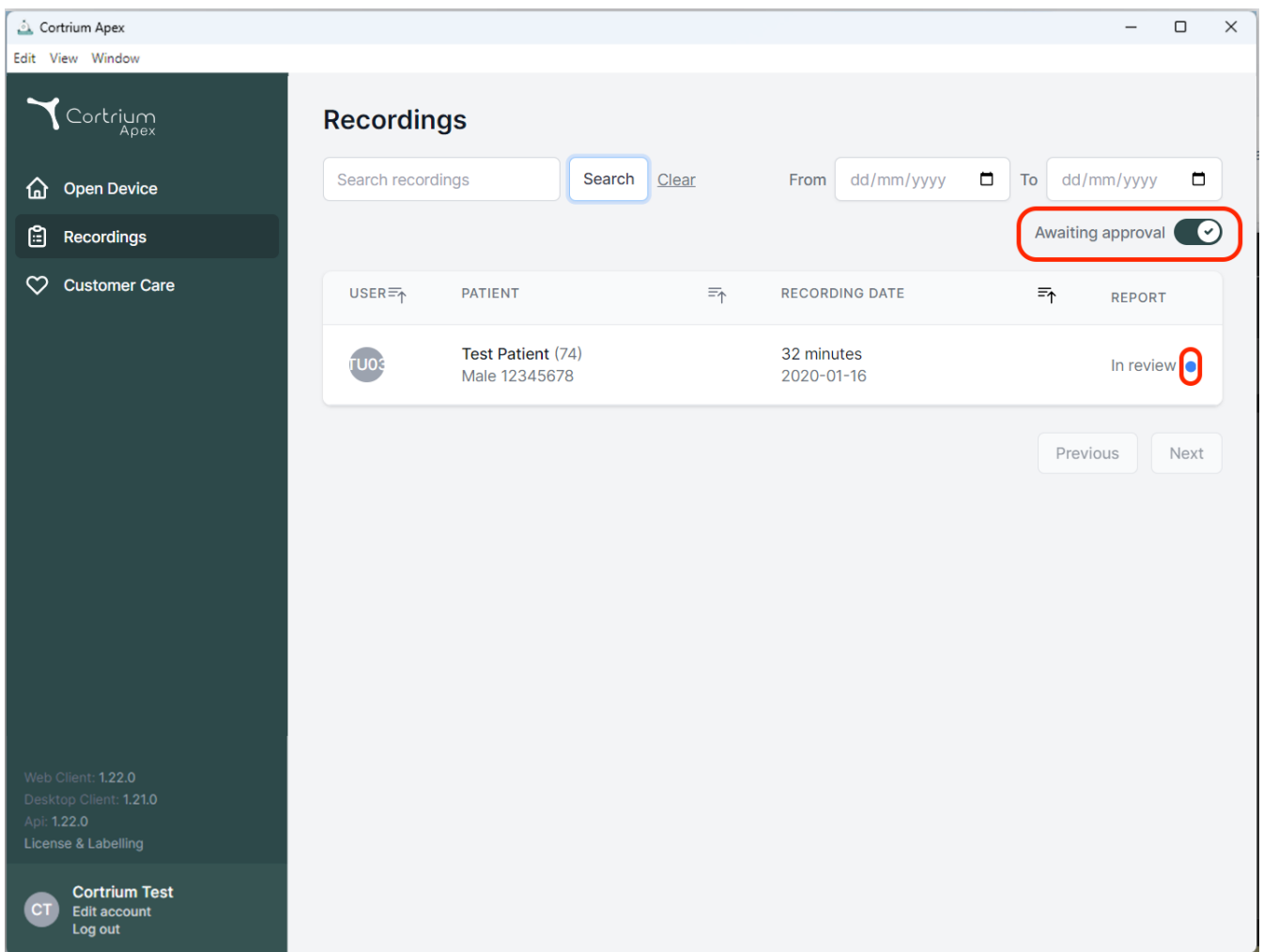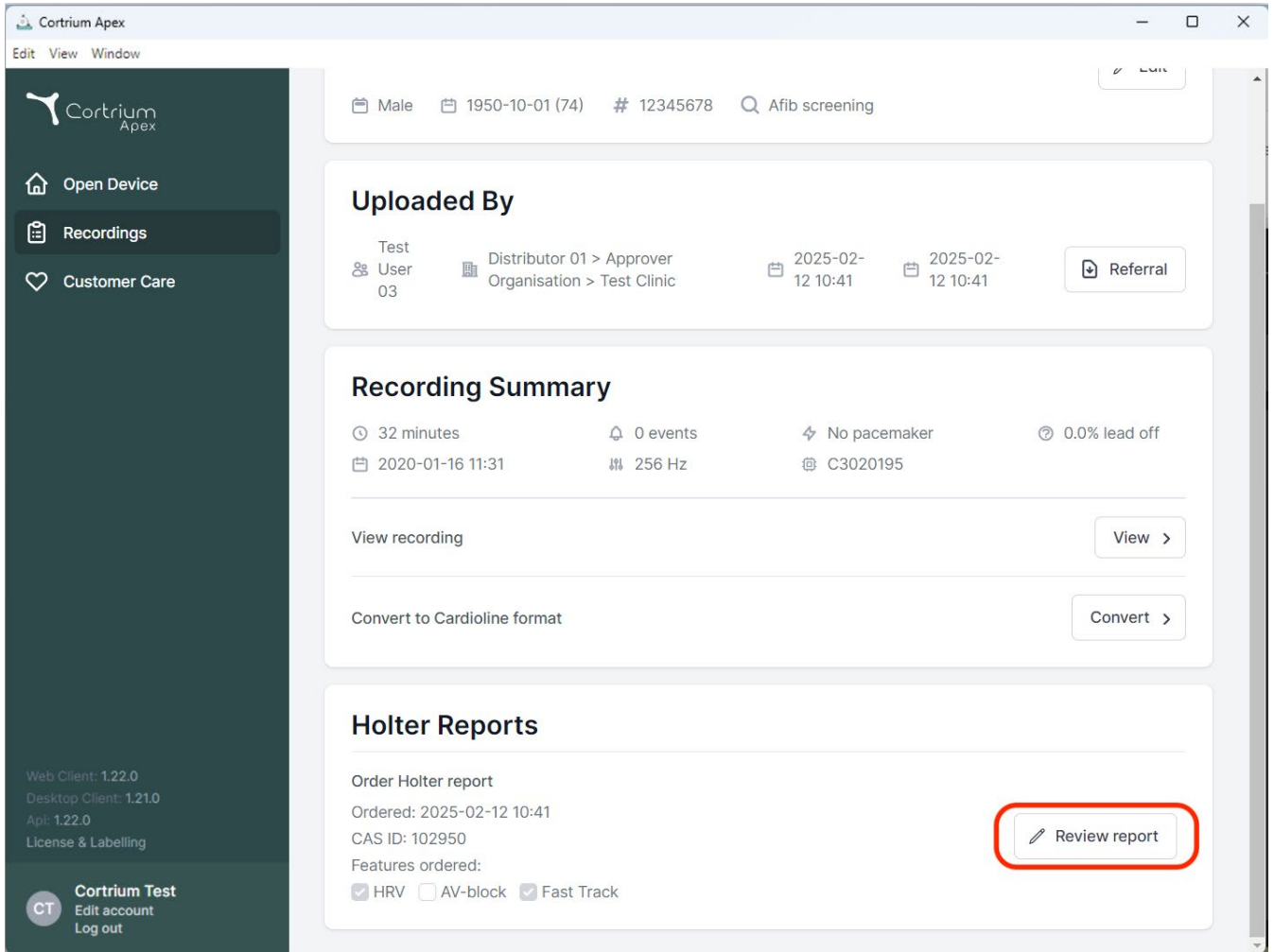
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, Afib screening. Below this is the "Uploaded By" section, showing Test User 03, Distributor 01 > Approver Organisation > Test Clinic, and dates 2025-02-12 10:41. A "Referral" button is present. The "Recording Summary" section displays: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, C3020195. Below this are buttons for "View recording" and "Convert to Cardioline format". The "Holter Reports" section shows "Order Holter report", "Ordered: 2025-02-12 10:41", "CAS ID: 102950", and "Features ordered: HRV, AV-block, Fast Track". A "Review report" button is highlighted with a red rectangle.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

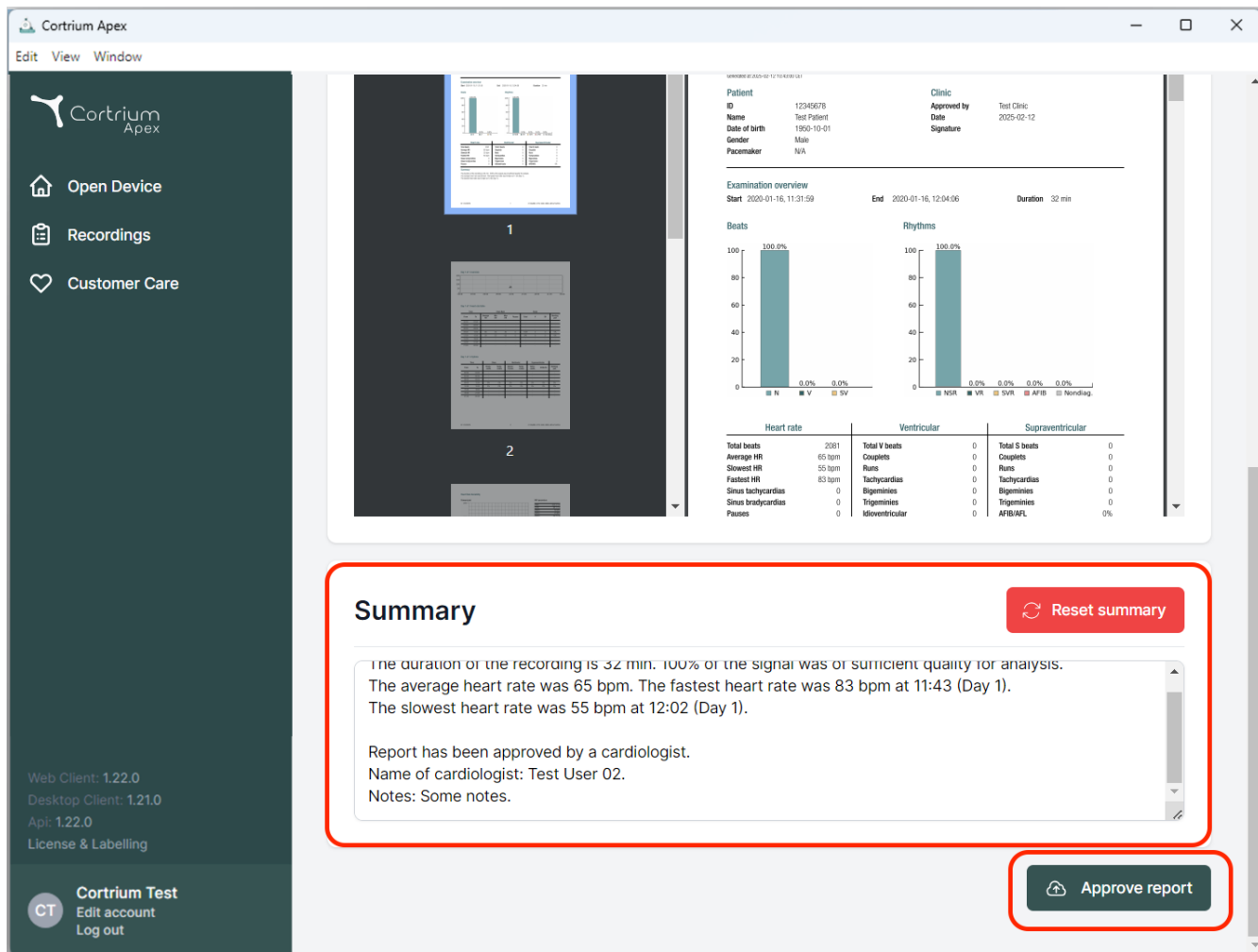
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

02/2025

„Cortrium ApS“ • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danija
El. paštas: info@cortrium.com • Interneto svetainė: www.cortrium.com

„Cortrium Apex“ UDI(01)05745000379033