Cortrium Apex 1.22

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Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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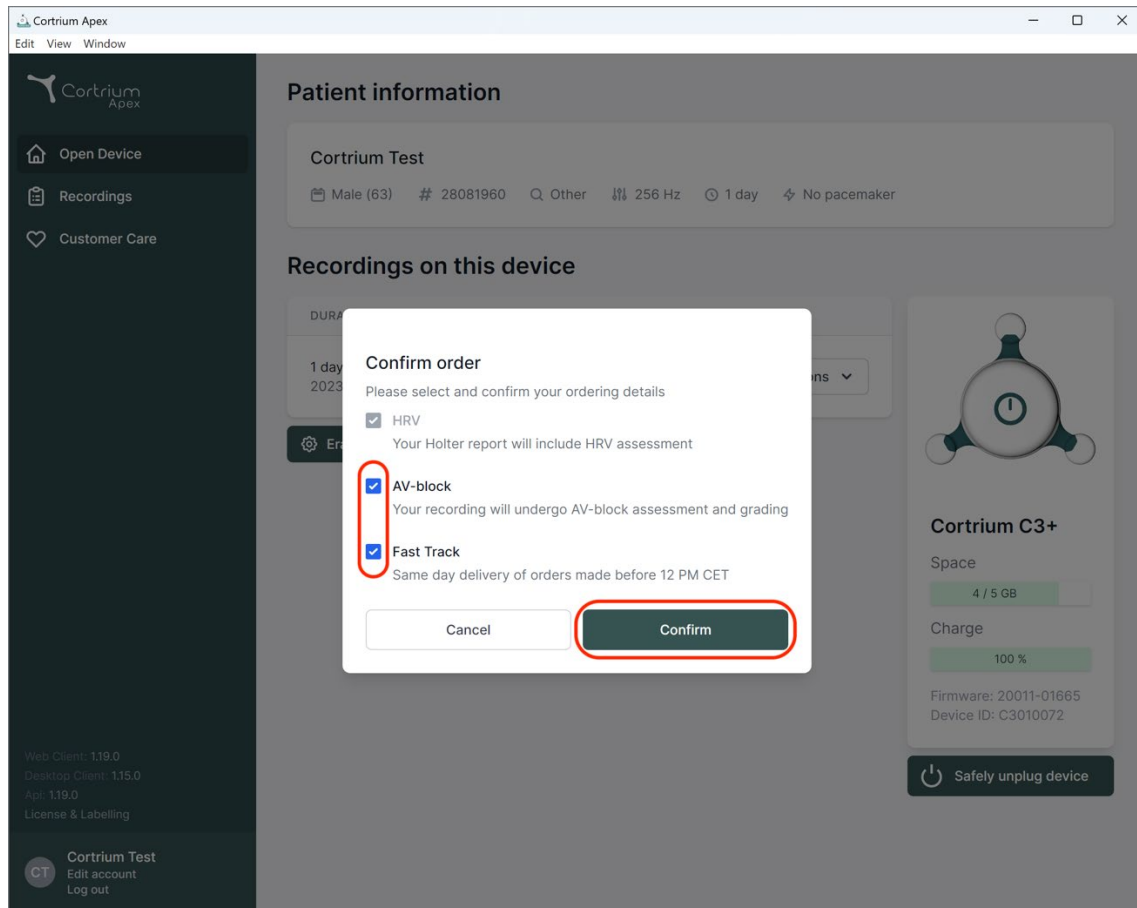
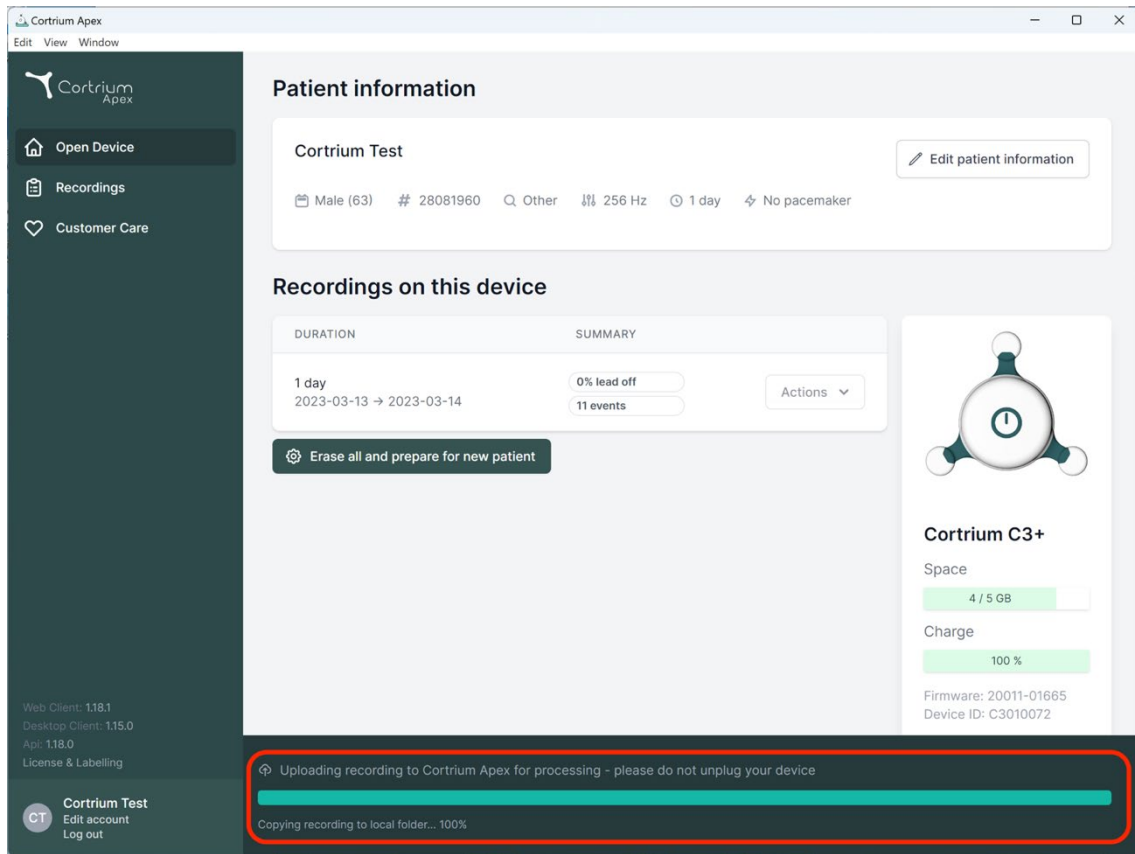
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and other parameters. Below this is a 'Recordings on this device' section with a table showing a 1-day recording period from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. On the right, there's a device status card for 'Cortrium C3+' showing 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar for 'Copying recording to local folder... 100%'.

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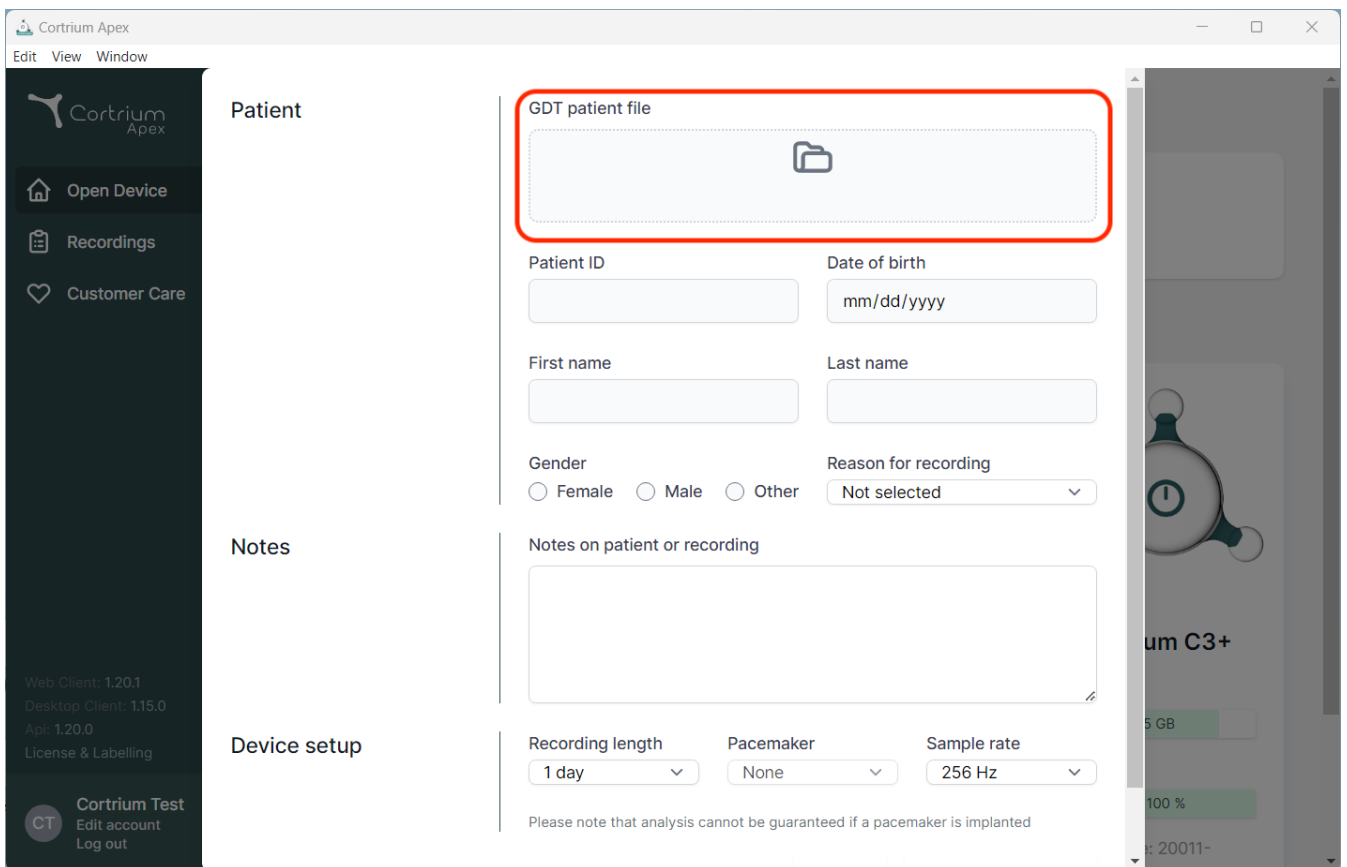
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup.

Patient Section:

- GDT patient file:** A red-bordered box containing a folder icon, indicating where to upload the patient file.
- Patient ID:** A text input field.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for Female, Male, and Other.
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

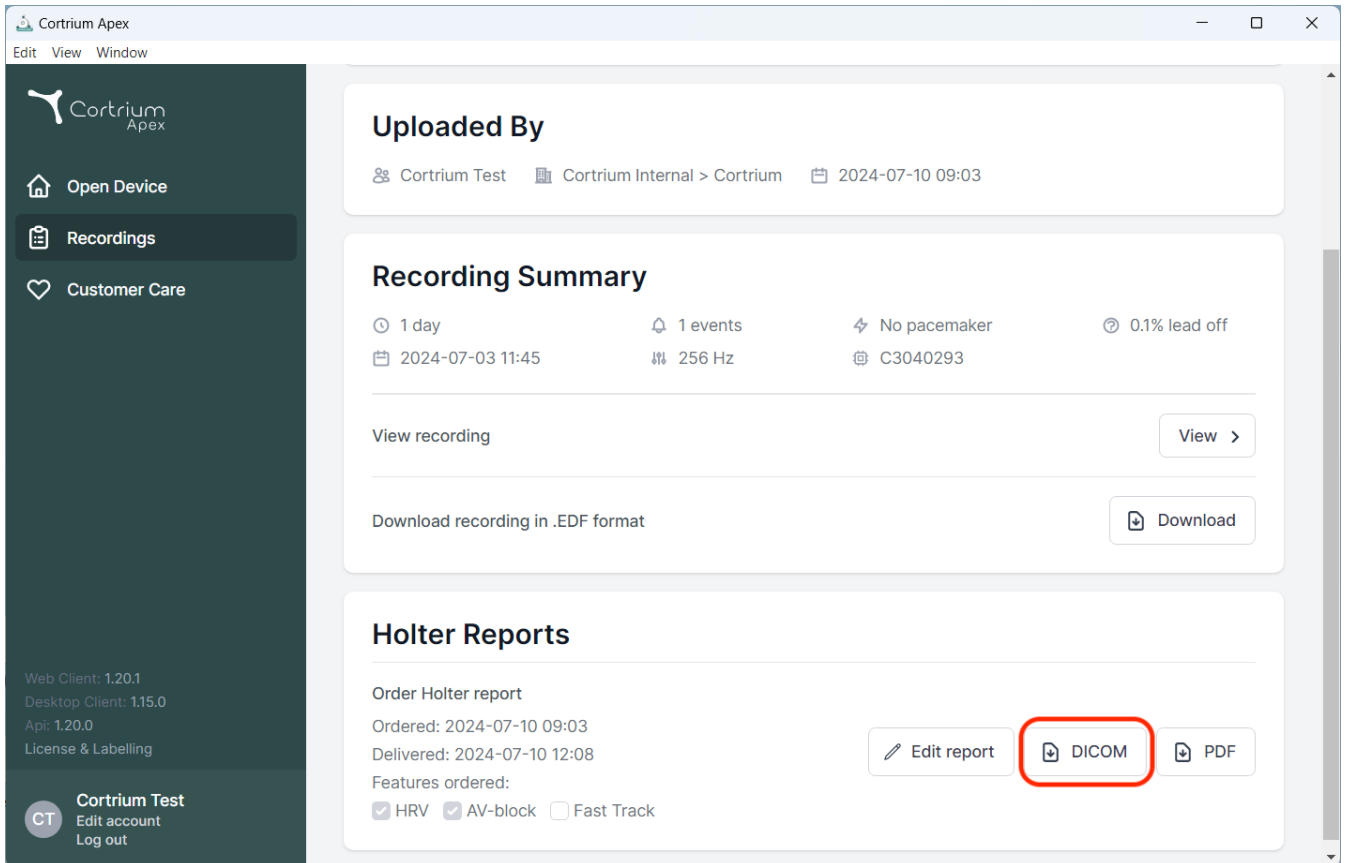
Device setup Section:

- Recording length:** A dropdown menu with "1 day" as the selected option.
- Pacemaker:** A dropdown menu with "None" as the selected option.
- Sample rate:** A dropdown menu with "256 Hz" as the selected option.

At the bottom of the Device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' details: 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red box), and 'PDF'.

The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a 'Cortrium Test' user profile with an 'Edit account' and 'Log out' option.

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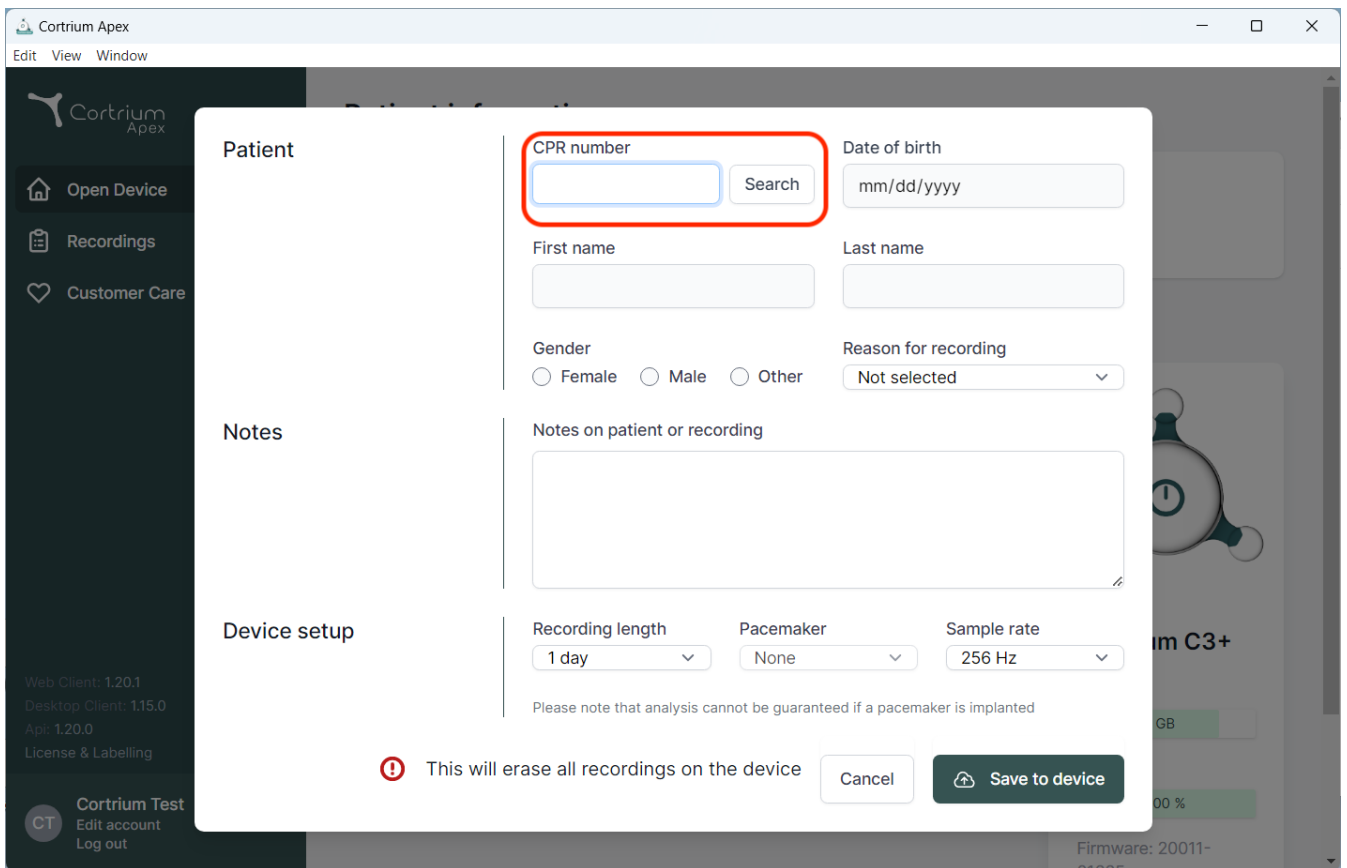
The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for searching and entering patient information. The "CPR number" field is highlighted with a red box, and a "Search" button is next to it. Other fields include "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), and "Notes on patient or recording". The "Device setup" section includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". Buttons for "Cancel" and "Save to device" are also present.

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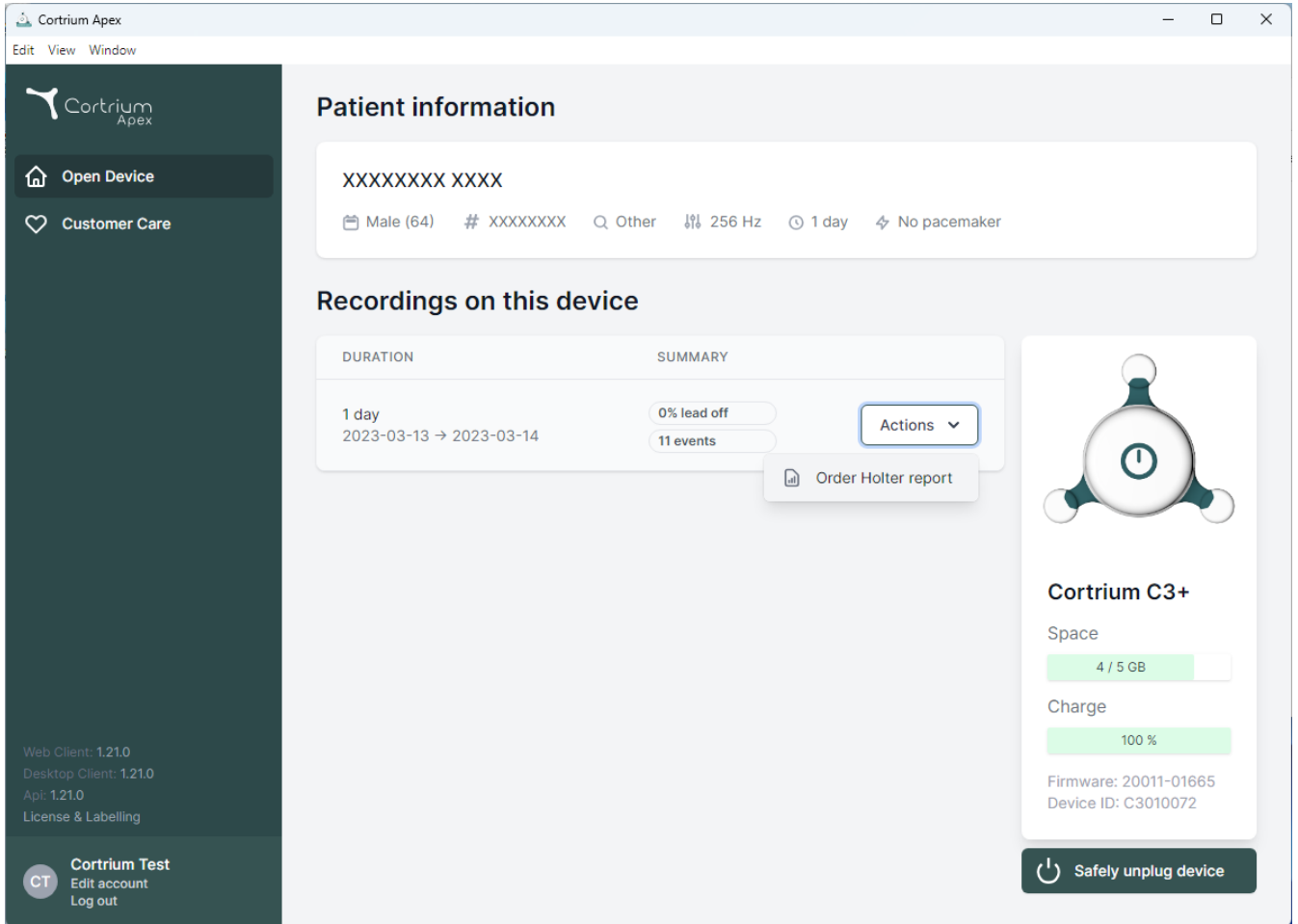The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: 'Open Device' and 'Customer Care'. The main content area is divided into sections for patient information and device recordings.

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Actions

Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

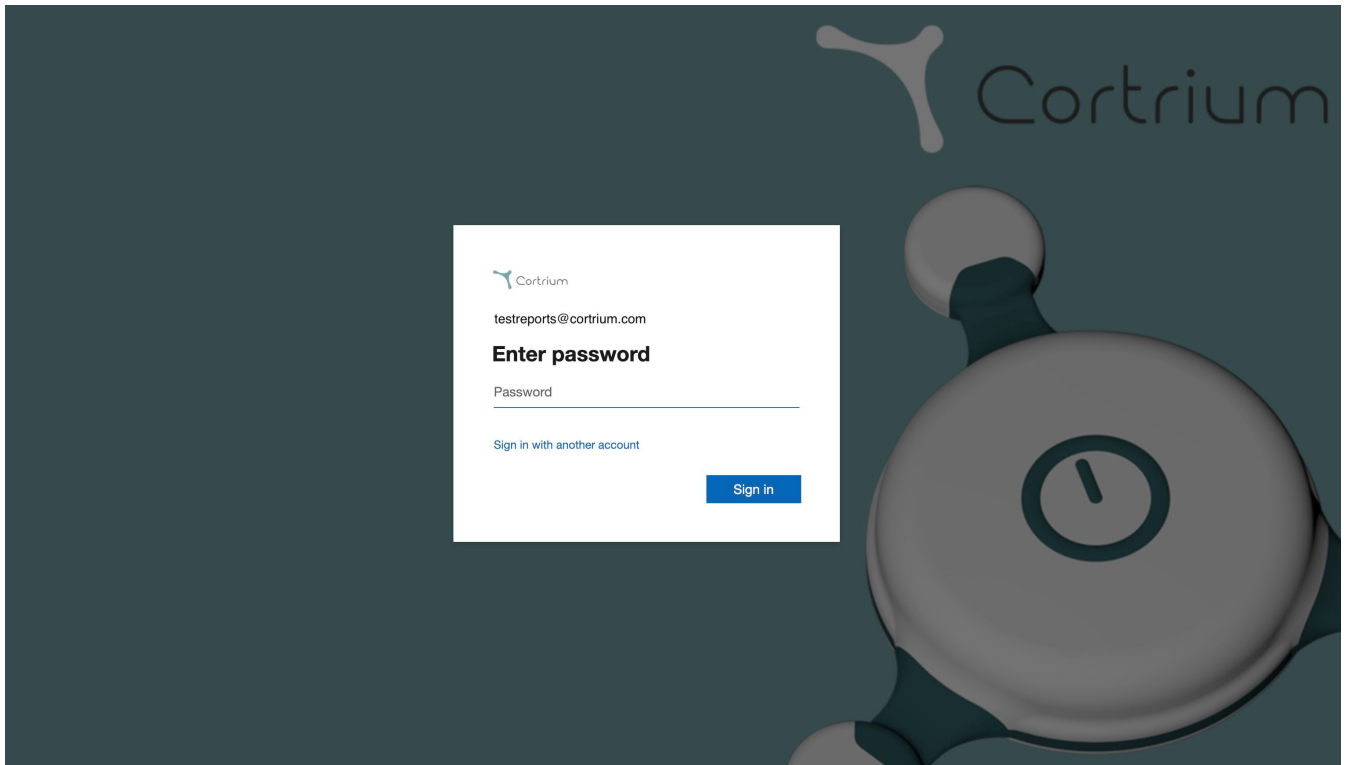
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Apt: 1.21.0
License & Labelling

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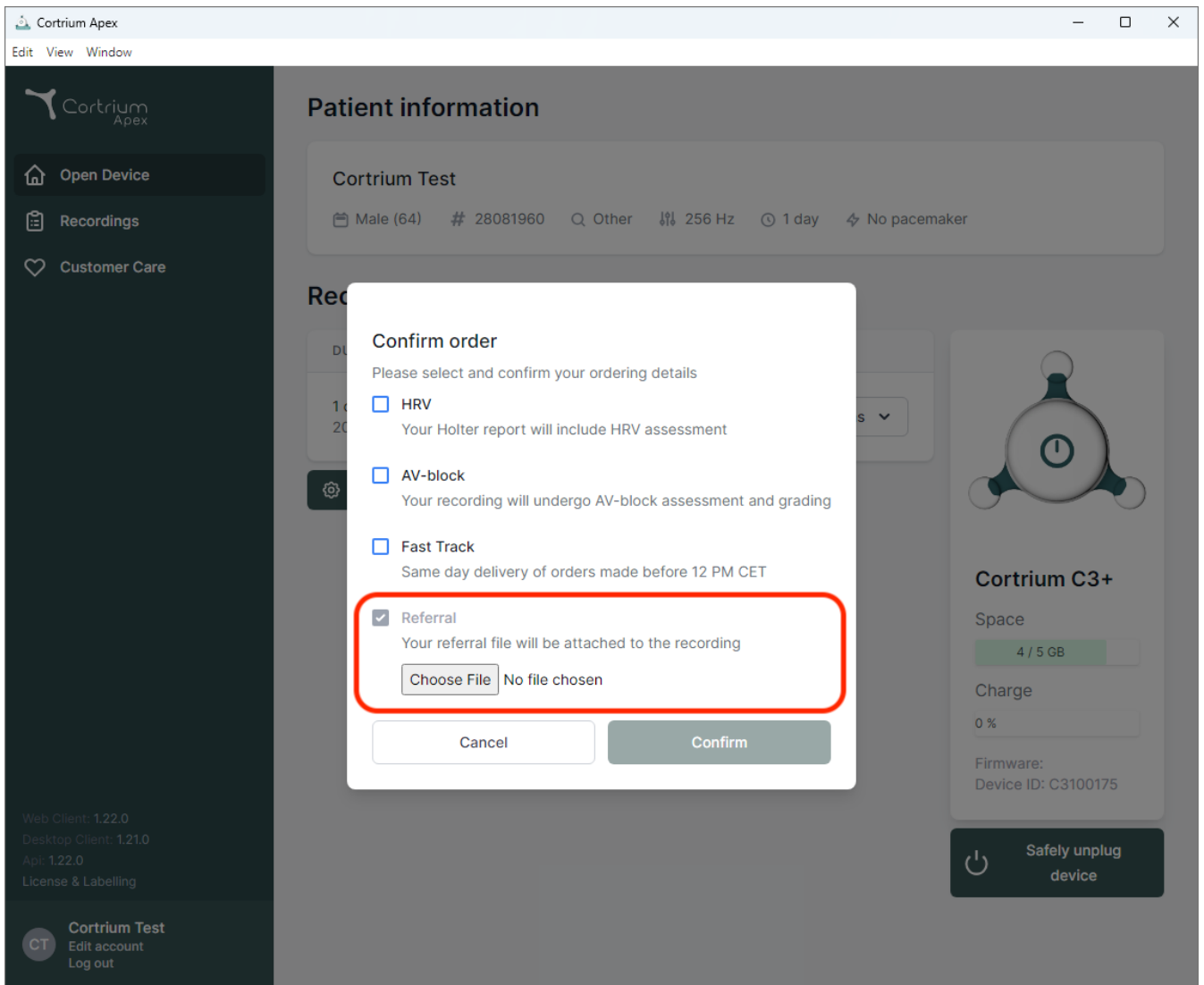
Confirm order

Please select and confirm your ordering details

- ☐ HRV
Your Holter report will include HRV assessment
- ☐ AV-block
Your recording will undergo AV-block assessment and grading
- ☐ Fast Track
Same day delivery of orders made before 12 PM CET
- ☒ Referral
Your referral file will be attached to the recording
 No file chosen

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
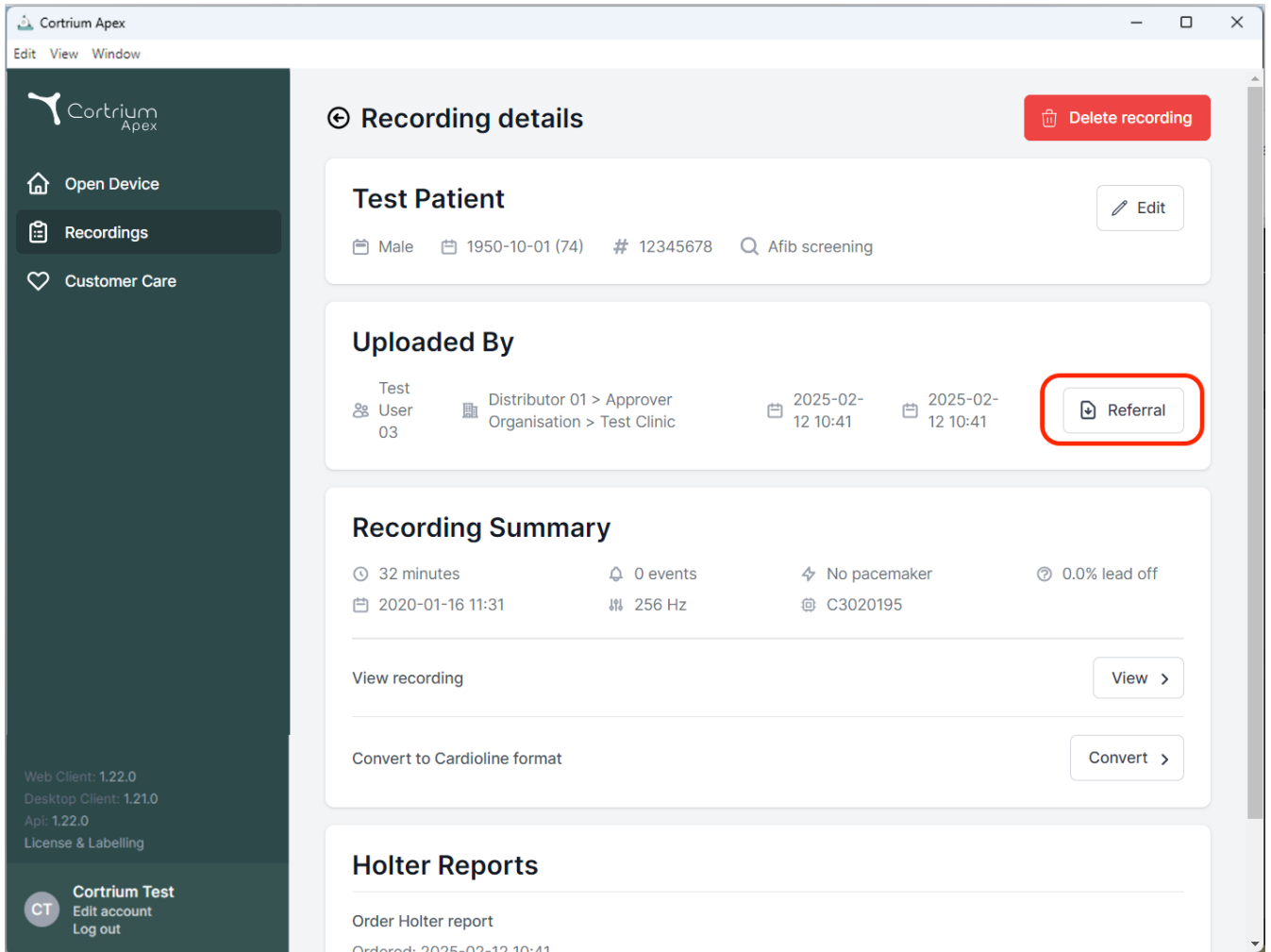
The screenshot displays the 'Recording details' page in the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into several sections:

- Recording details:** Includes a 'Delete recording' button.
- Test Patient:** Displays patient information (Male, 1950-10-01 (74), # 12345678, Afib screening) and an 'Edit' button.
- Uploaded By:** Shows the user 'Test User 03' and the distributor 'Distributor 01 > Approver Organisation > Test Clinic'. It also includes upload dates '2025-02-12 10:41' and a 'Referral' button highlighted with a red box.
- Recording Summary:** Provides key metrics: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date '2020-01-16 11:31'. It also shows '256 Hz' and 'C3020195'. Below this are buttons for 'View recording' and 'Convert to Cardioline format'.
- Holter Reports:** Includes an 'Order Holter report' button and a note 'Ordered: 2025-02-12 10:41'.

The sidebar also contains version information: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', and 'License & Labelling'. At the bottom, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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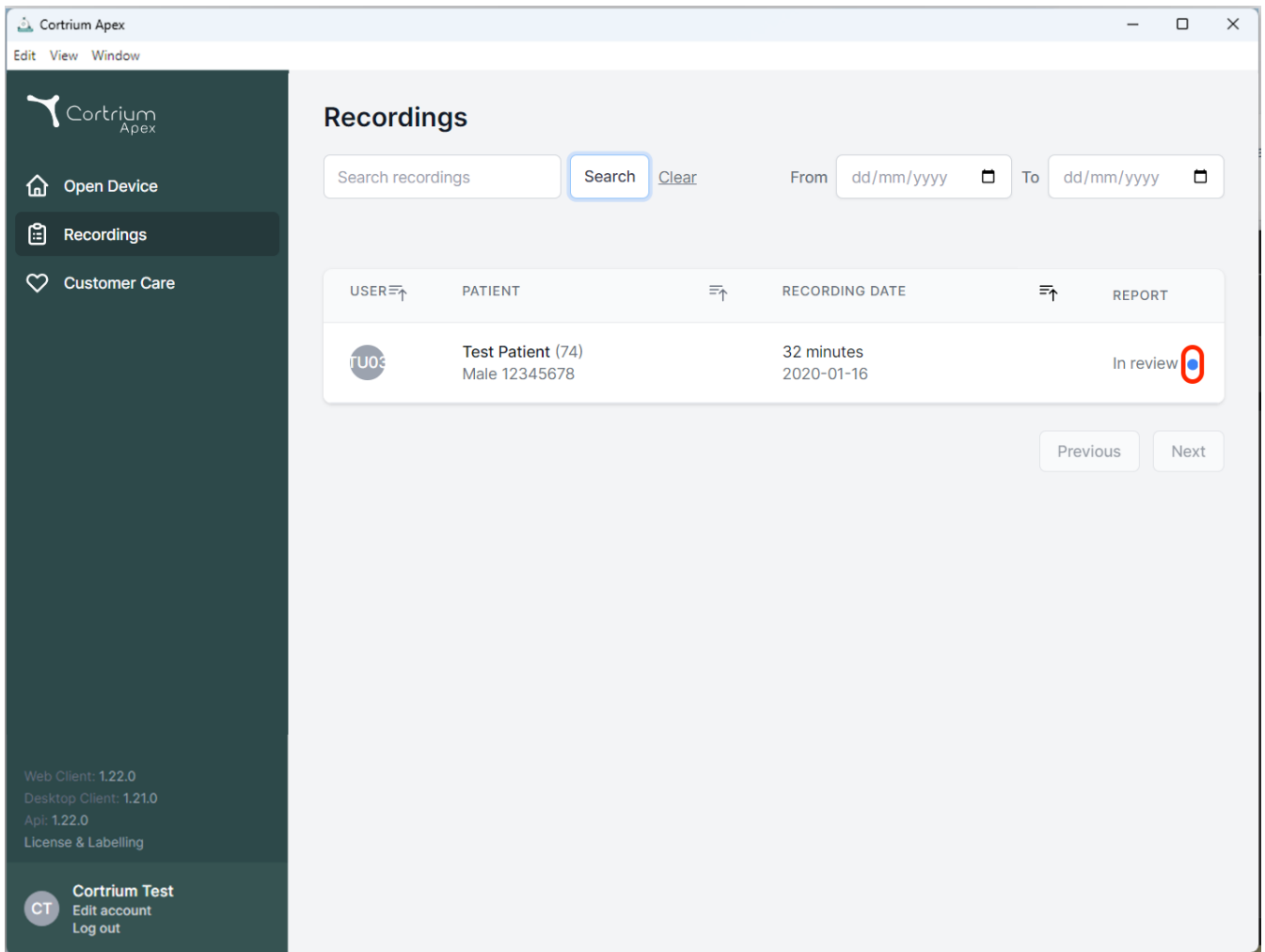Edit View Window

Recordings

Search recordings **Search** [Clear](#) From dd/mm/yyyy To dd/mm/yyyy

USER	PATIENT	RECORDING DATE	REPORT
U03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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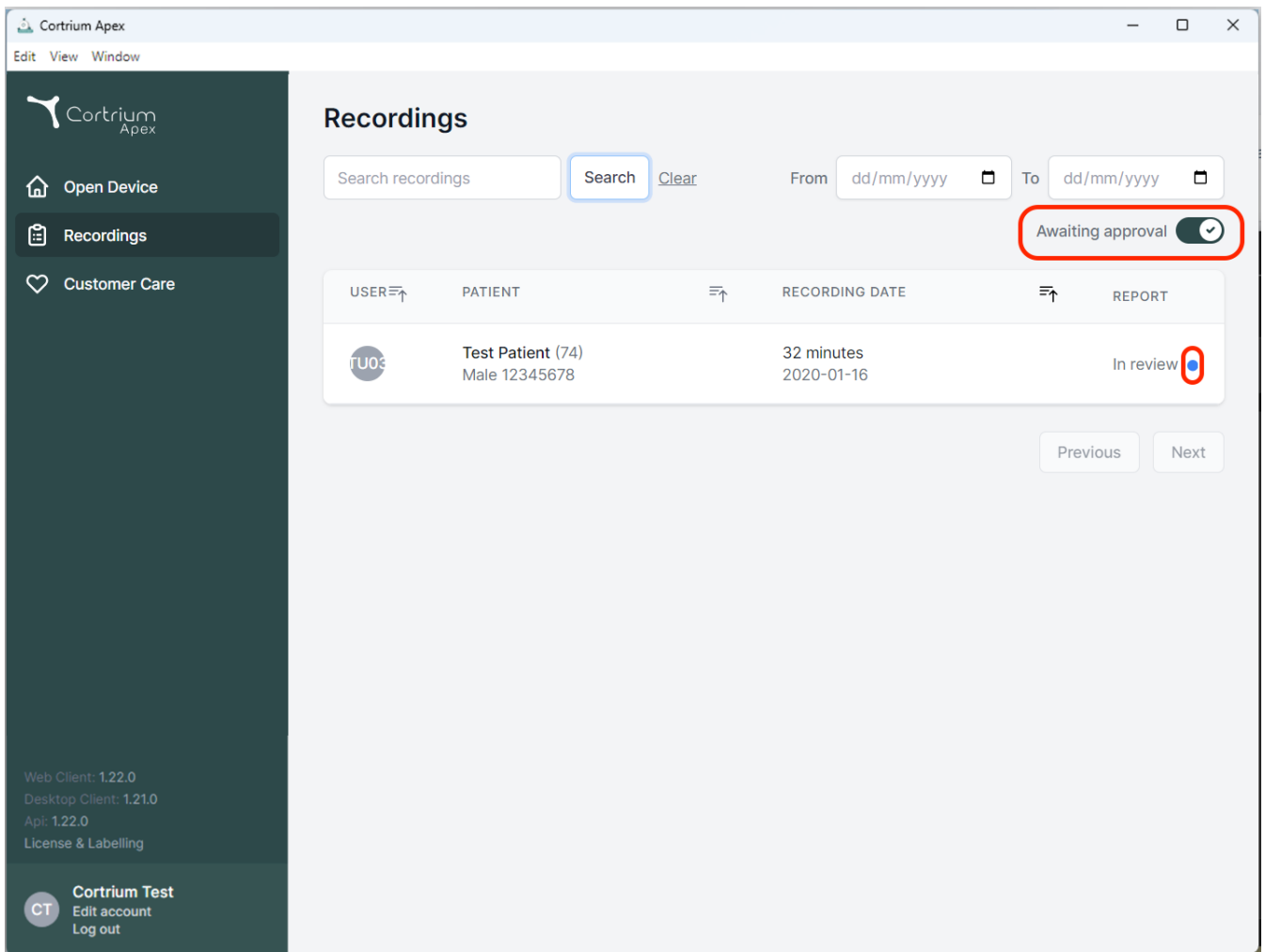
Recordings

Search recordings From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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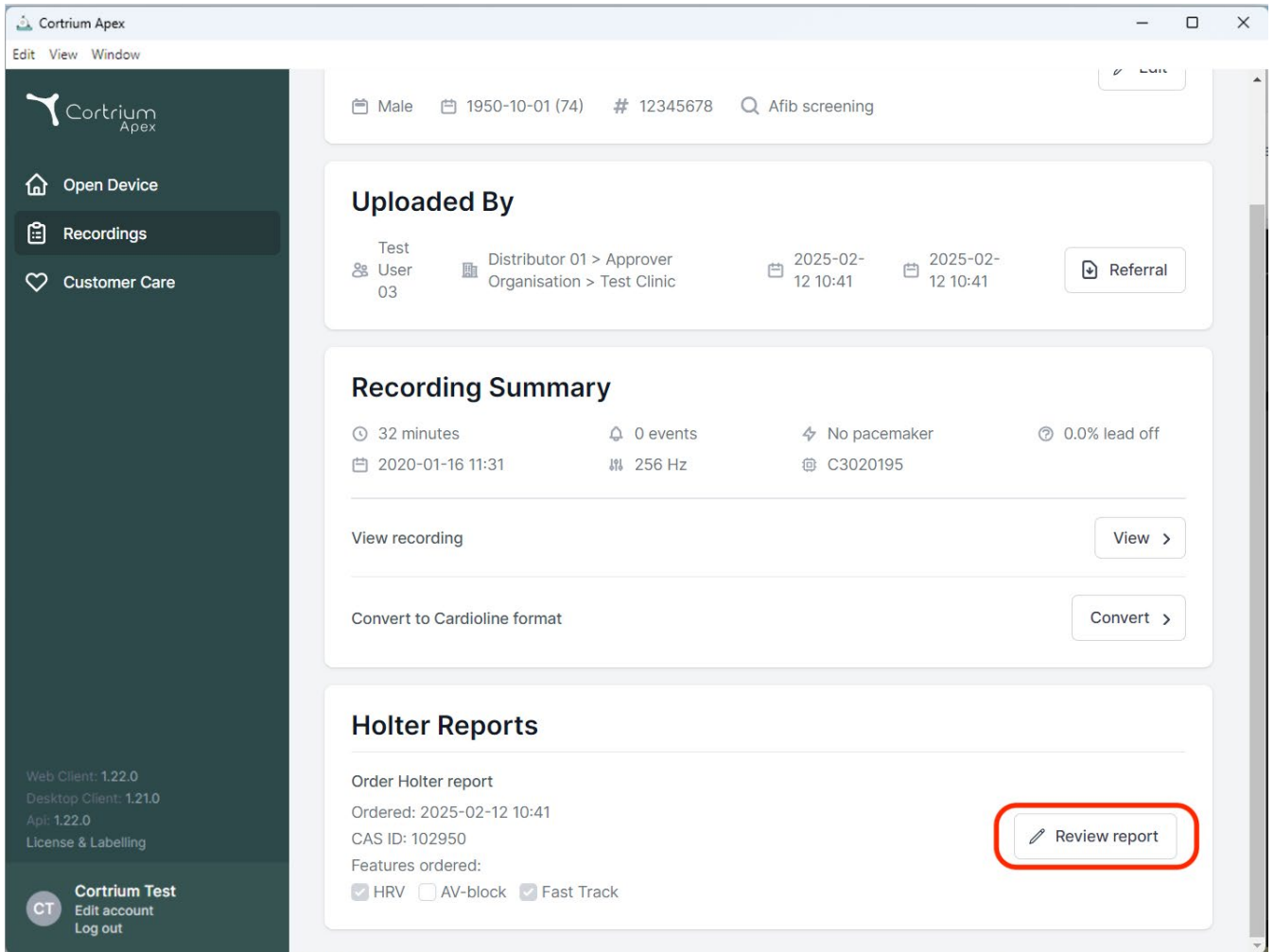
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes the Cortrium Apex logo, a menu with 'Edit', 'View', and 'Window', and a search bar. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver), and organization (Organisation > Test Clinic). It also includes dates (2025-02-12 10:41) and a 'Referral' button.
- Recording Summary:** A section providing details about the recording, including duration (32 minutes), events (0 events), pacemaker status (No pacemaker), and lead off percentage (0.0% lead off). It also shows the recording date (2020-01-16 11:31) and frequency (256 Hz). Below this, there are buttons for 'View recording' and 'Convert to Cardioline format'.
- Holter Reports:** A section for ordering and reviewing holter reports. It shows the order date (2025-02-12 10:41), CAS ID (102950), and features ordered (HRV, AV-block, Fast Track). A red box highlights the 'Review report' button.

At the bottom left, there is a 'Cortrium Test' section with links for 'Edit account' and 'Log out'.

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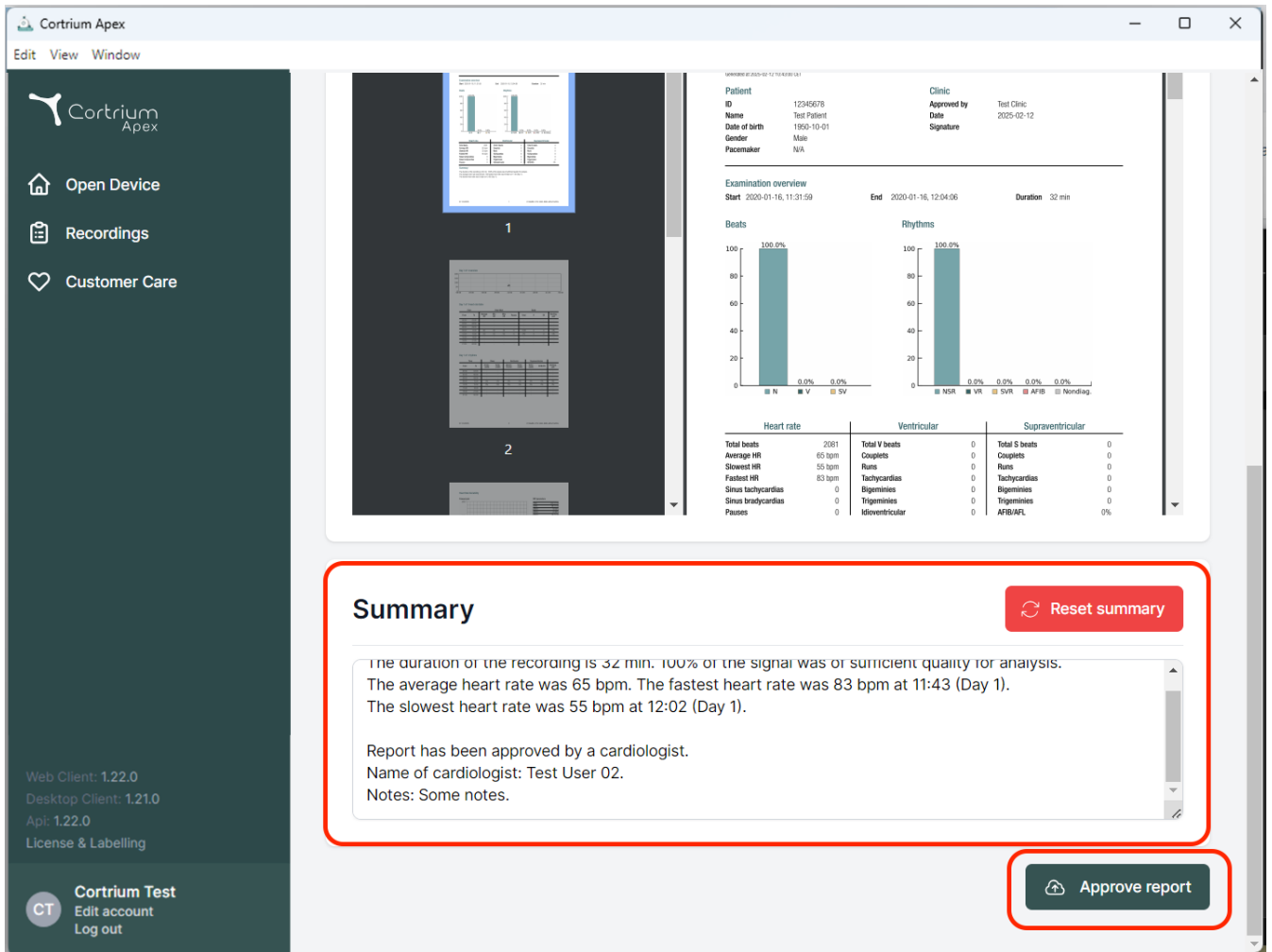Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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