Cortrium Apex 1.26

Cortrium Apex 1.26

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
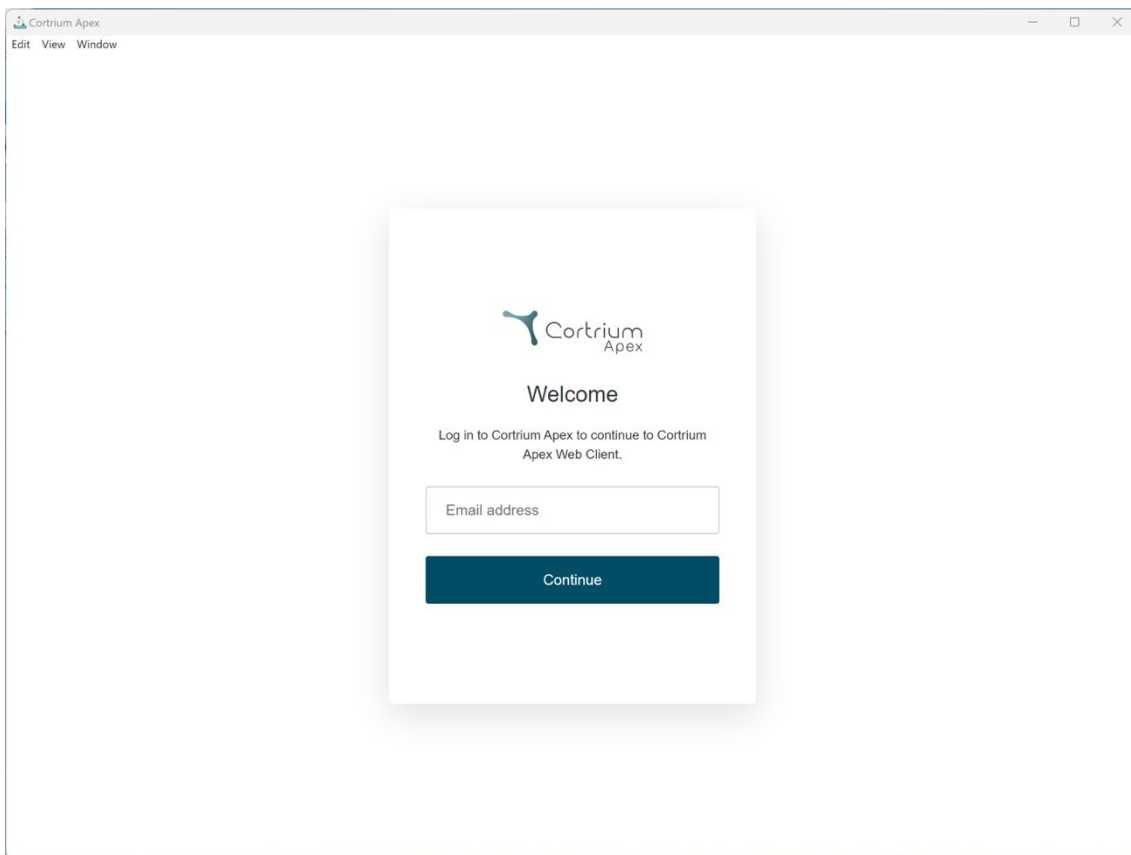
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle shadow. At the top of the form is the Cortrium Apex logo. Below the logo is the heading "Welcome". Underneath the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." followed by a text input field labeled "Email address". At the bottom of the form is a dark blue button with the text "Continue" in white.

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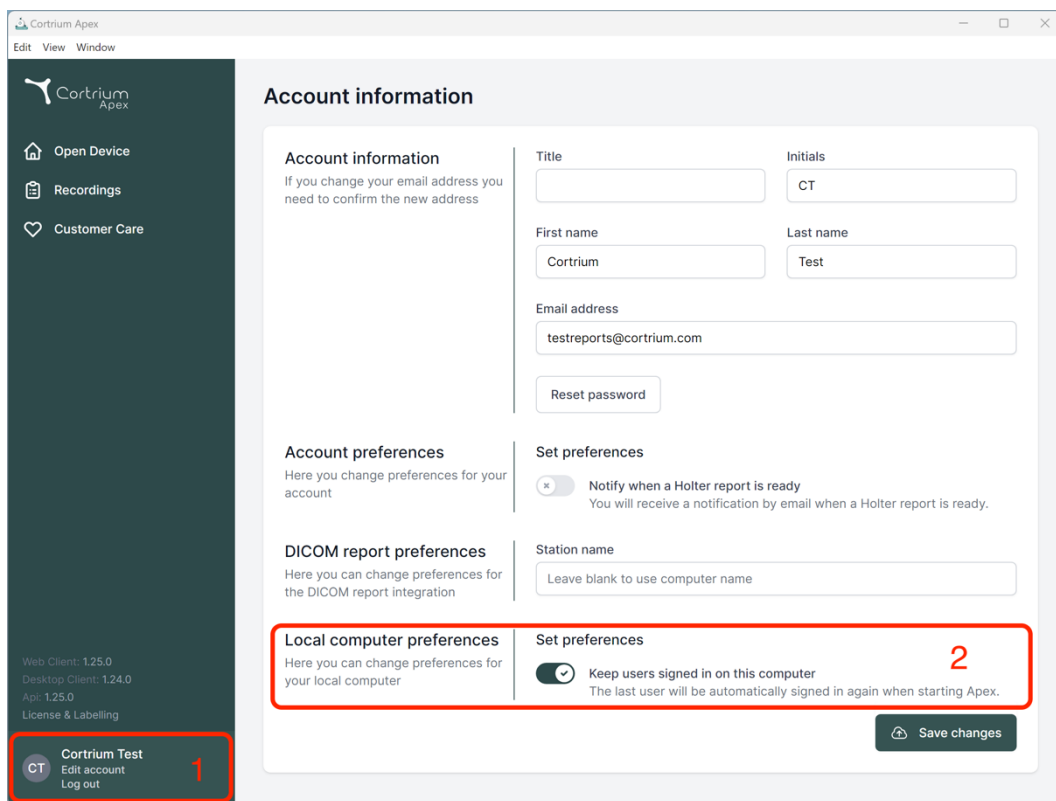The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.25.0', 'Desktop Client: 1.24.0', 'Api: 1.25.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out', marked with a red '1'.

The main content area is titled 'Account information'. It contains several sections:

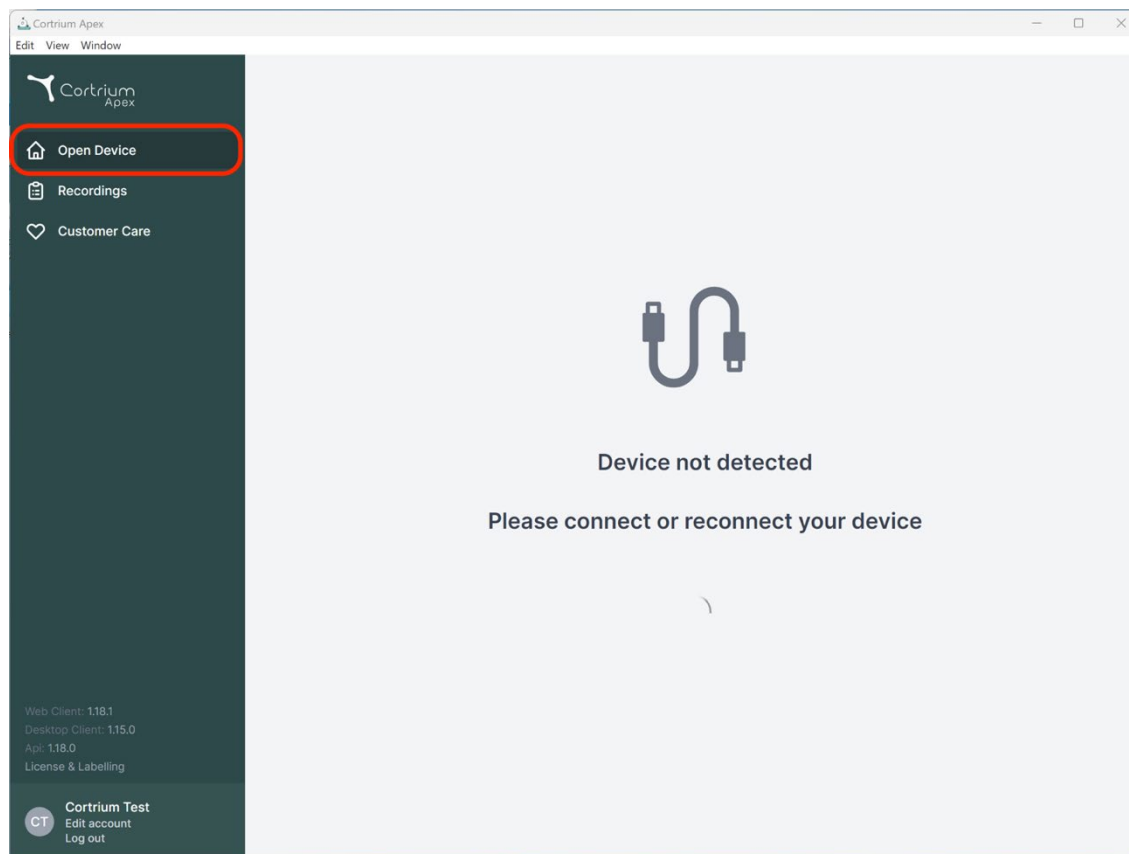
- Account information:** Fields for Title (empty), Initials (CT), First name (Cortrium), Last name (Test), and Email address (testreports@cortrium.com). There is a 'Reset password' button.
- Account preferences:** A section for changing account preferences.
- DICOM report preferences:** A section for changing preferences for DICOM report integration.
- Local computer preferences:** A section for changing preferences for the local computer, marked with a red box and a red '2'. It includes a toggle switch for 'Keep users signed in on this computer' which is currently turned on. Below the toggle, it says 'The last user will be automatically signed in again when starting Apex.'
- Set preferences:** A section for setting preferences, including a 'Station name' field with the placeholder 'Leave blank to use computer name'.

At the bottom right of the main content area is a 'Save changes' button.

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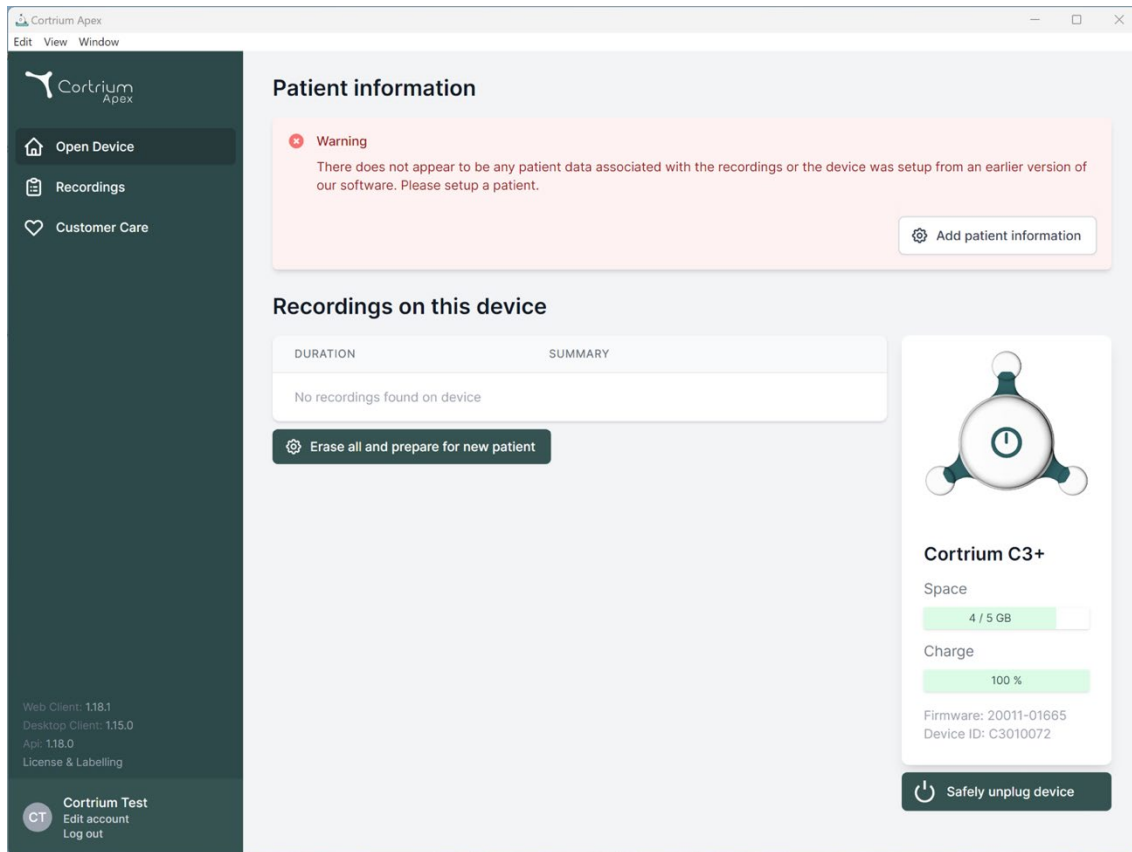
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning This will erase all recordings on the device

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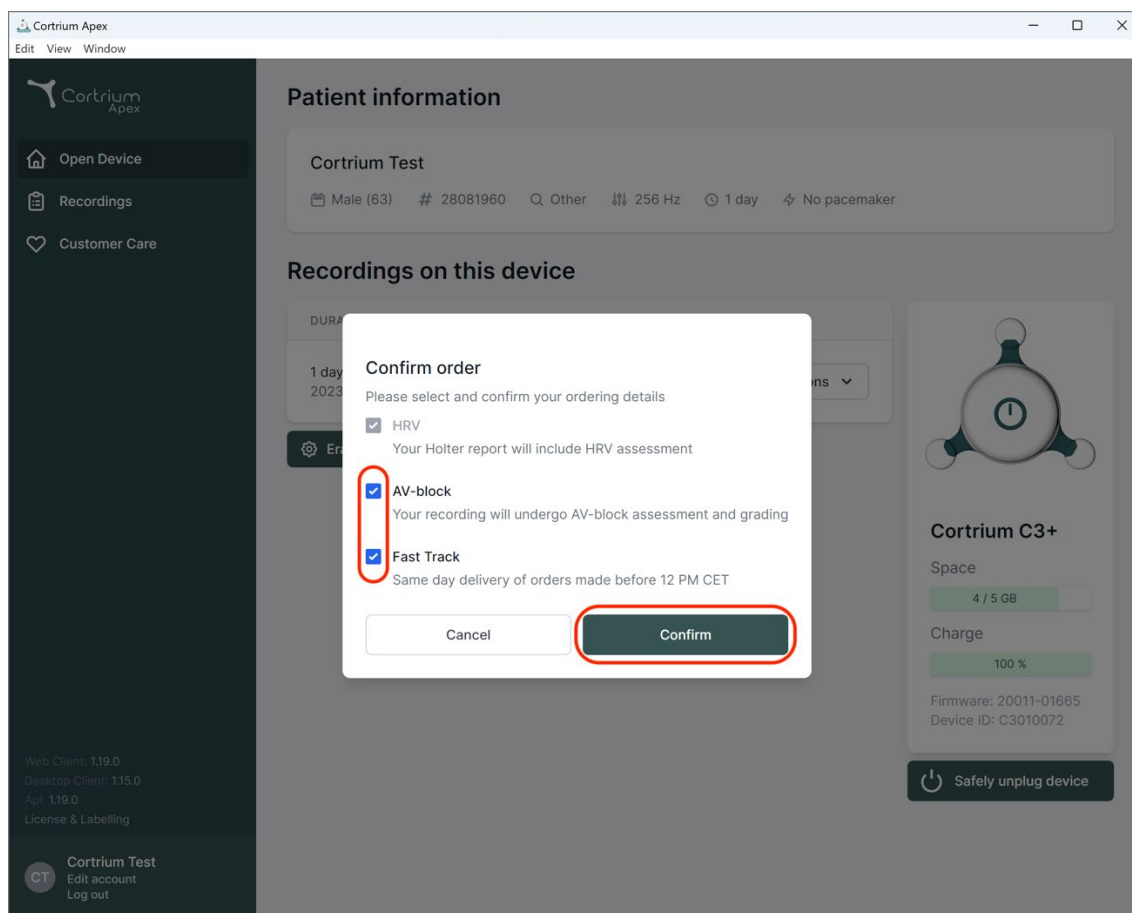
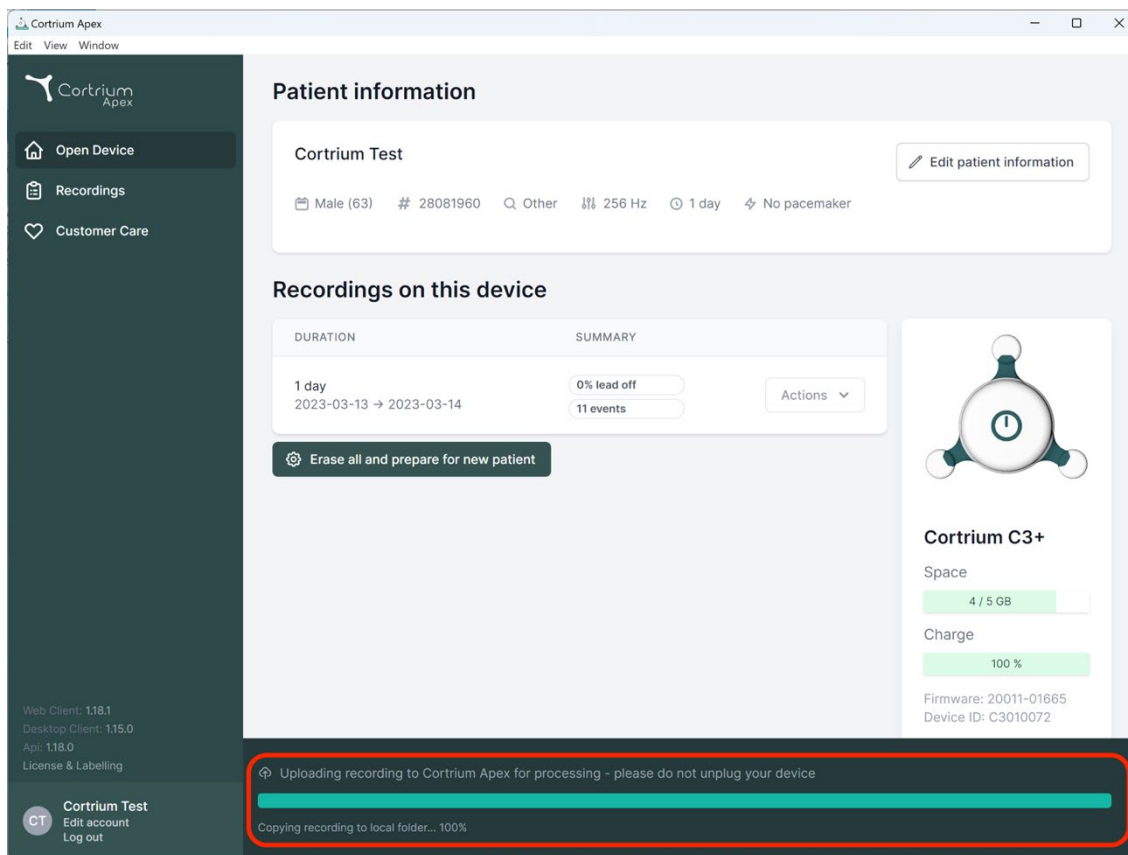
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and recording duration (1 day). Below this, the 'Recordings on this device' section shows a table with columns for 'DURATION' and 'SUMMARY'. The table lists a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. To the right, a device status card for 'Cortrium C3+' shows 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar for 'Copying recording to local folder... 100%'. The bottom left of the sidebar shows 'Cortrium Test' with 'Edit account' and 'Log out' options.

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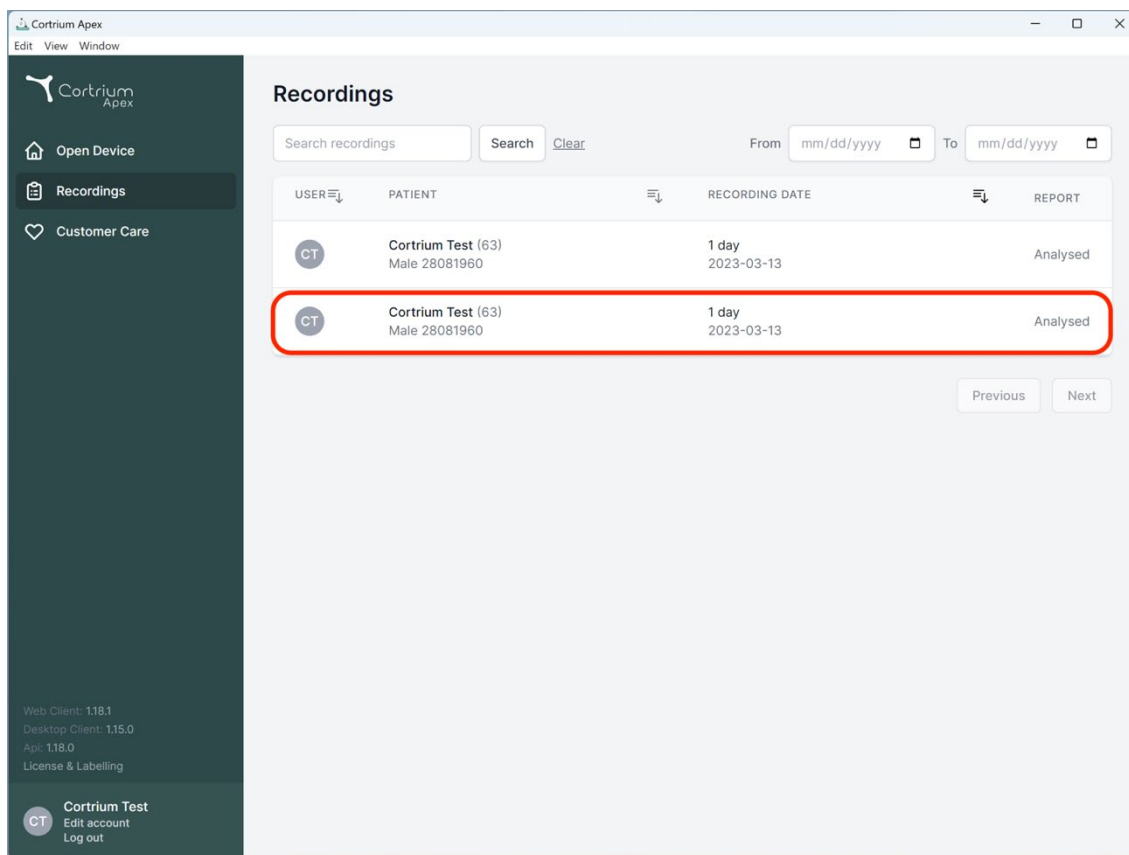
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. At the bottom of the sidebar, it shows 'Web Client: 118.1', 'Desktop Client: 115.0', 'App: 118.0', 'License & Labelling', and a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main area has a title 'Recordings' and a search bar. Below the search bar is a table with the following data:

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The second row of the table is highlighted with a red border. At the bottom right of the table area are 'Previous' and 'Next' buttons.

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
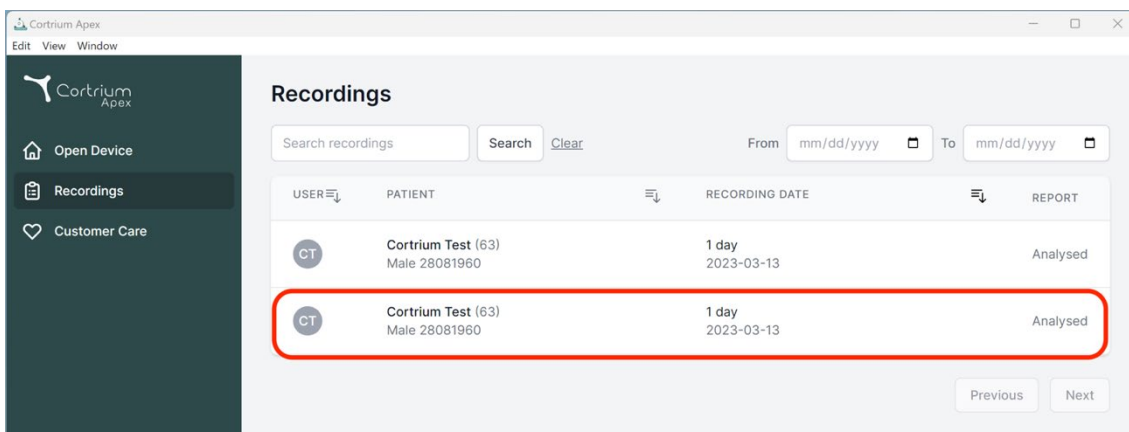
The screenshot shows the 'Recordings' section of the Cortrium Apex software. It features a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area displays a table of recordings with columns for 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. Two identical rows are shown, both for 'Cortrium Test (63)' and 'Male 28081960', recorded on '2023-03-13' for a duration of '1 day'. The status is 'Analysed'. The second row is highlighted with a red rectangle. Search and date range filters are visible at the top of the table.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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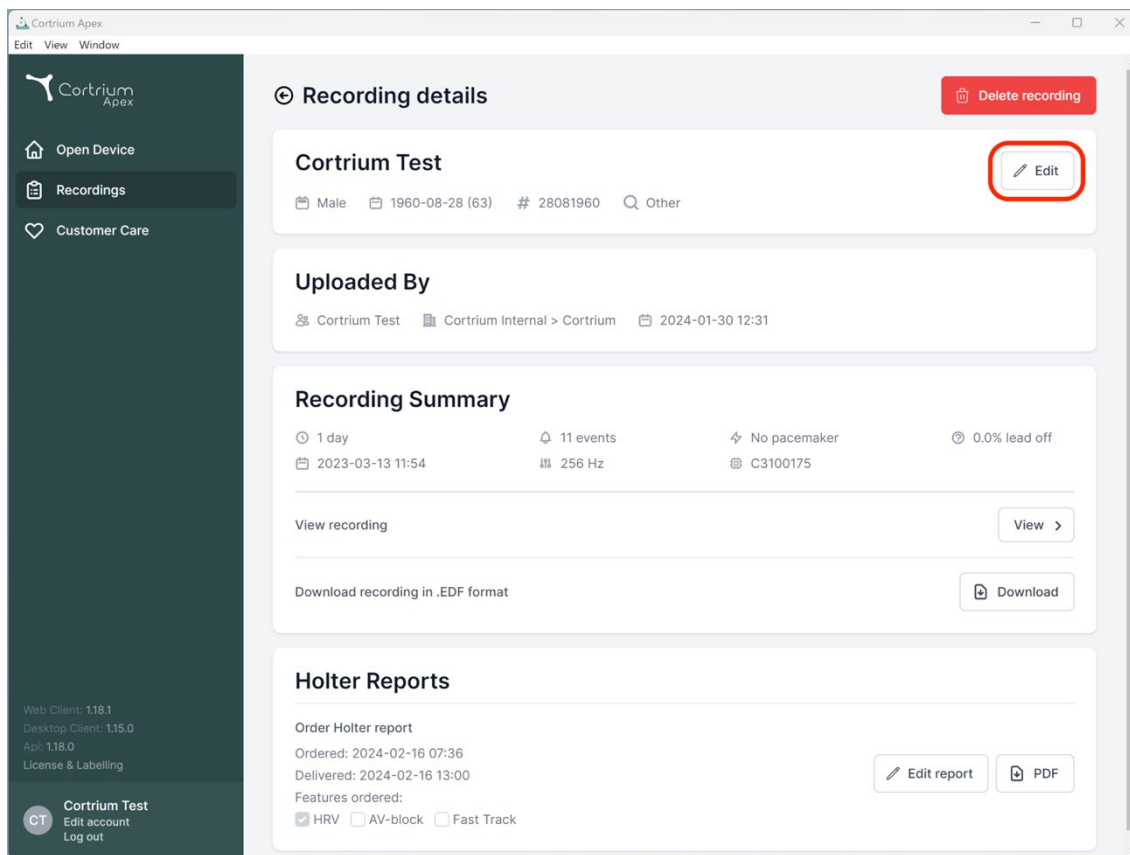Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

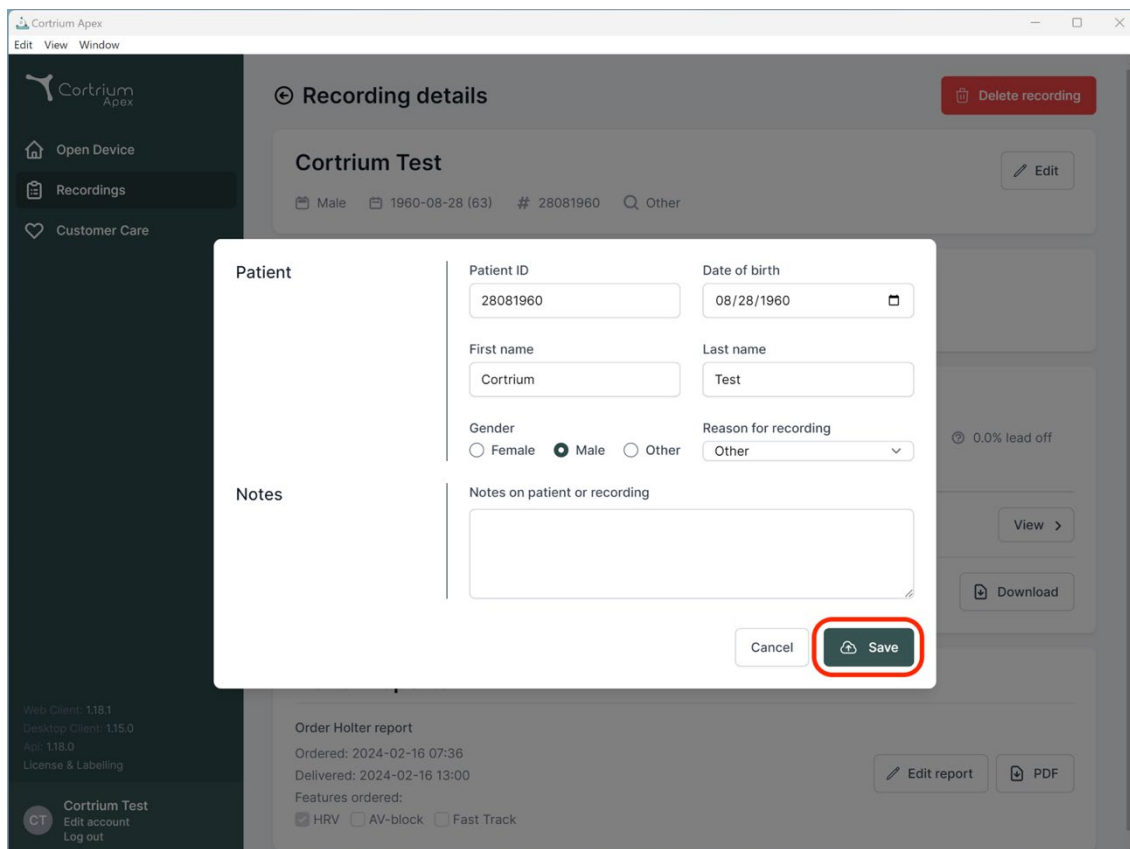
Order Holter report

Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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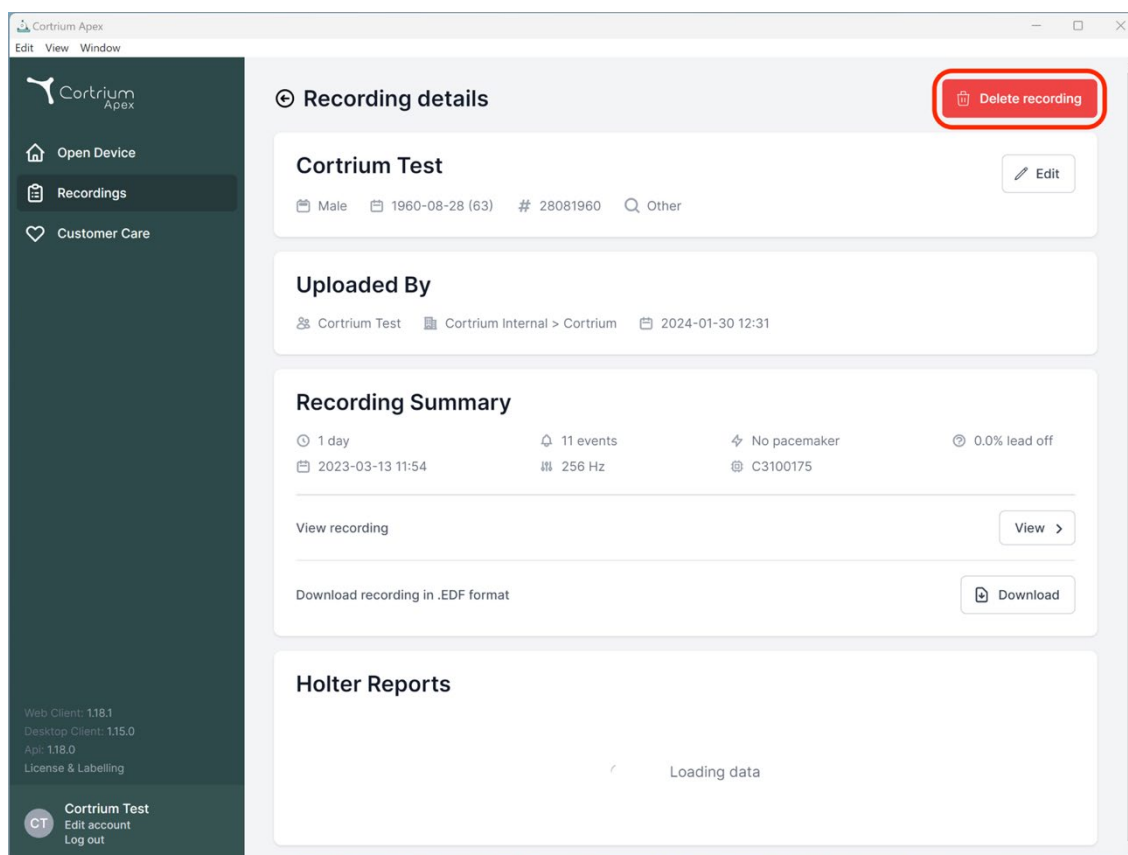
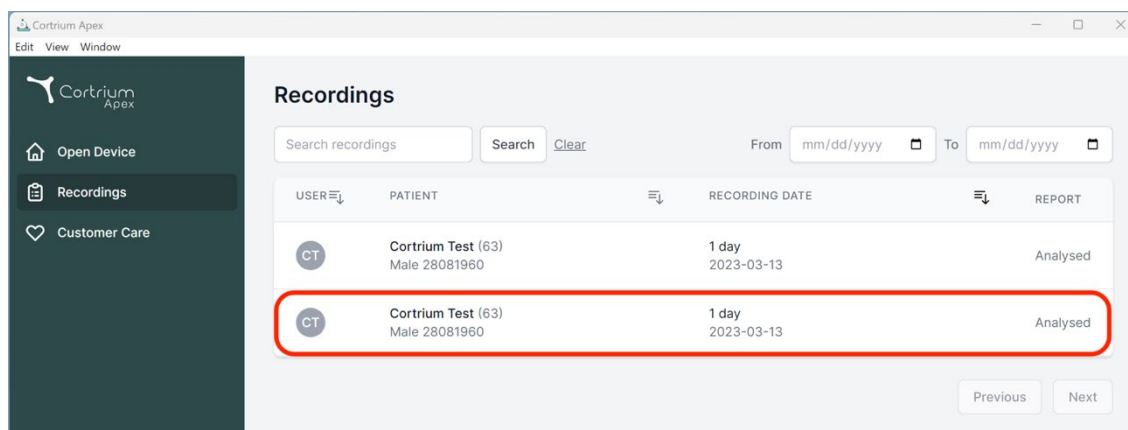
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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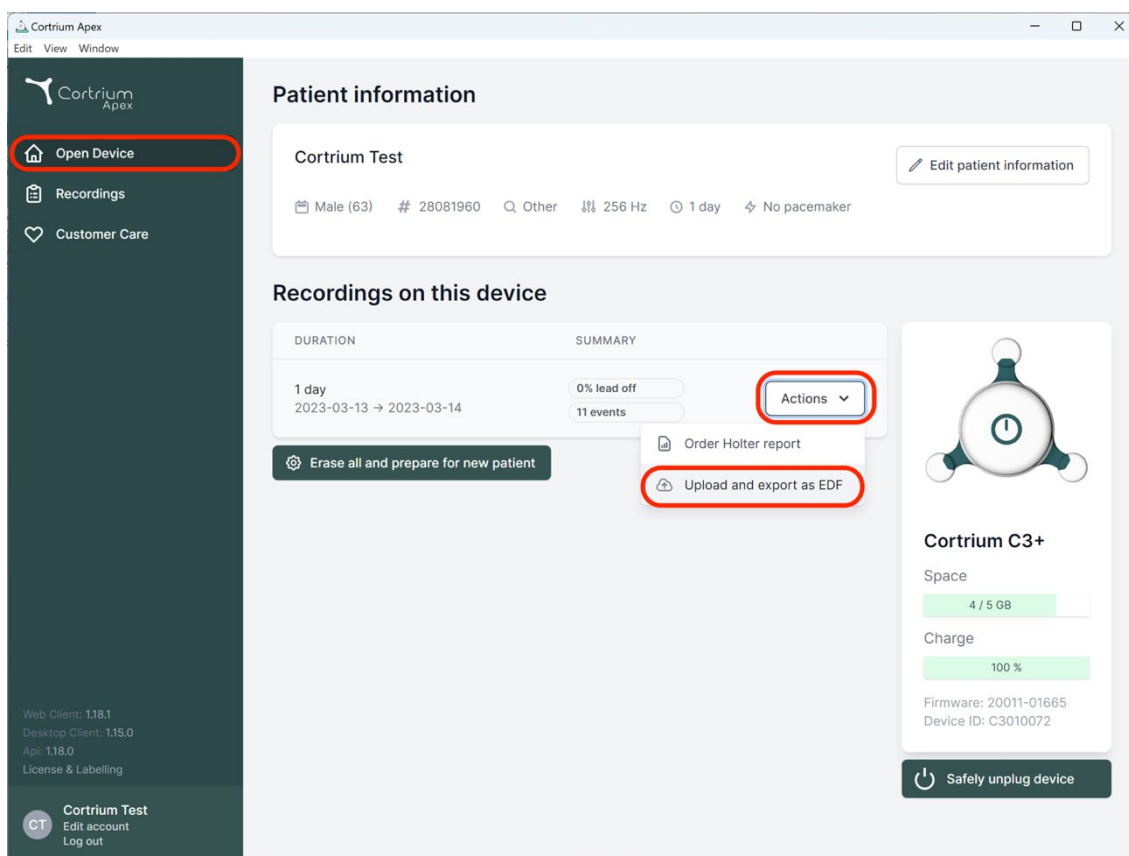
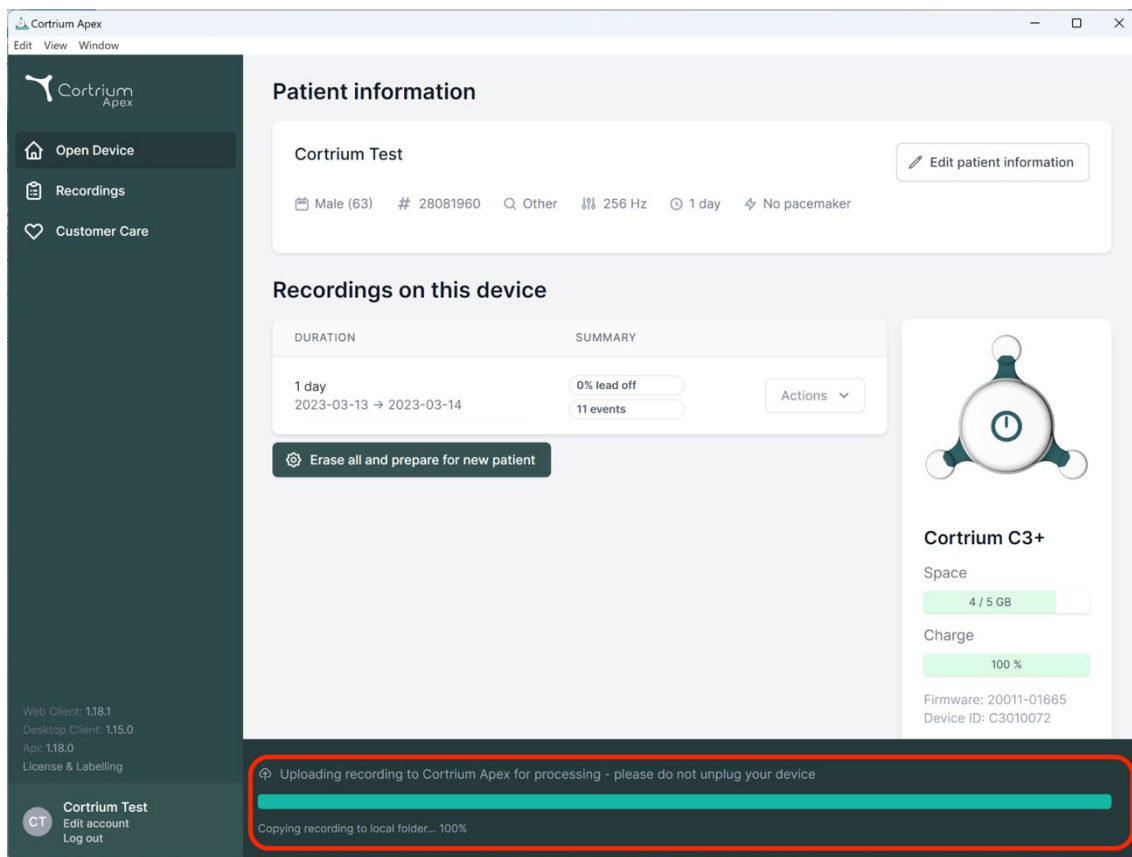
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

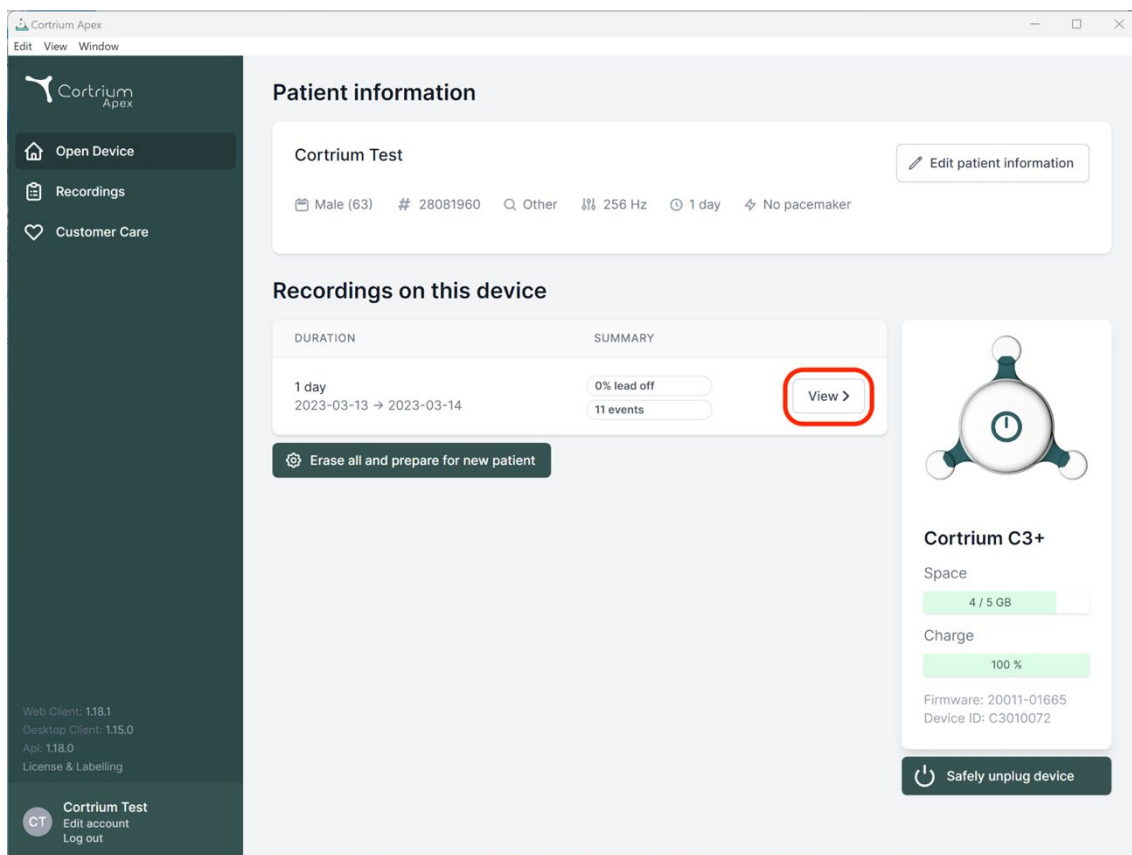
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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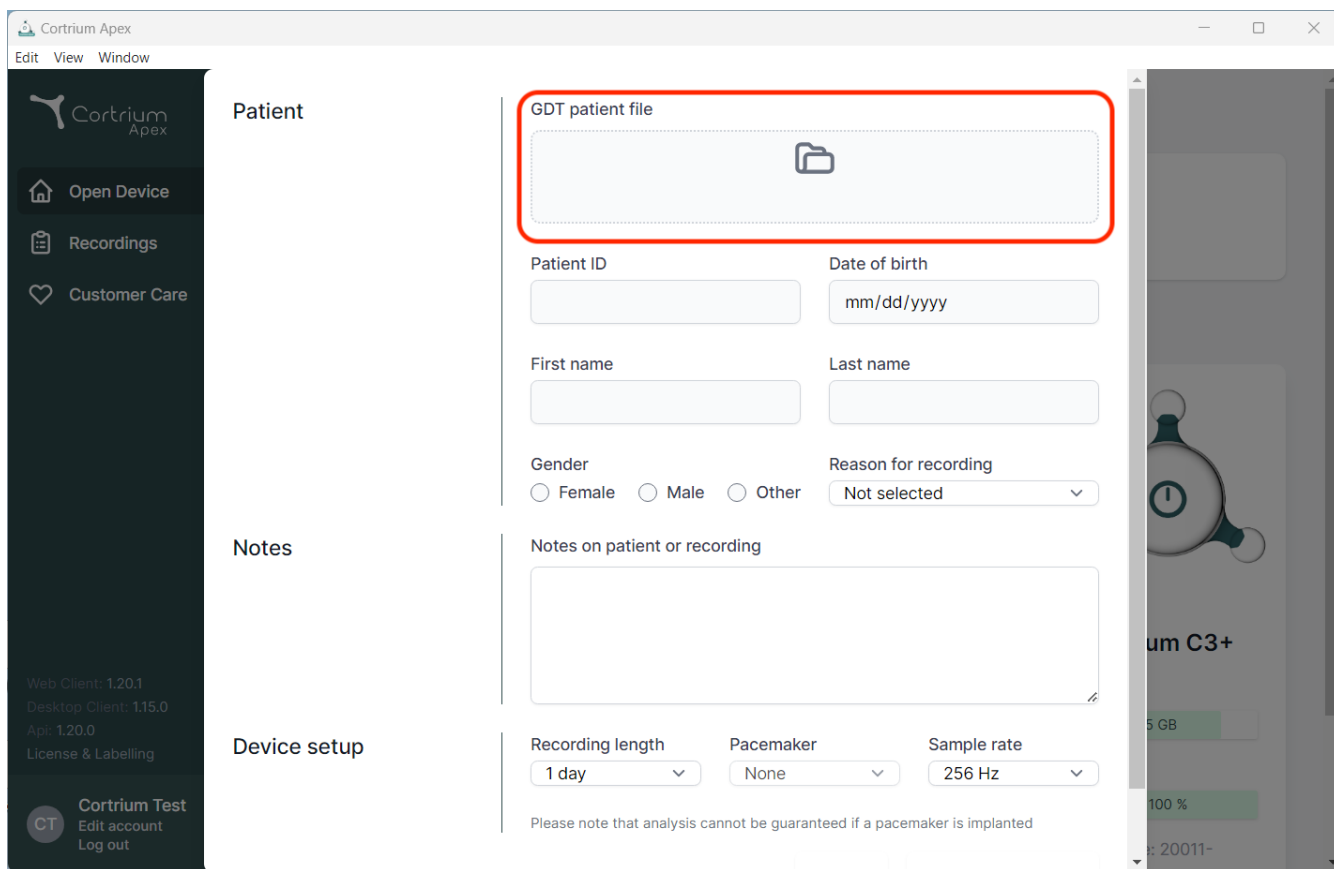
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

Patient section:

- GDT patient file:** A red-bordered box containing a folder icon, indicating where to upload the patient file.
- Patient ID:** A text input field.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes section:

- Notes on patient or recording:** A large text area for entering notes.

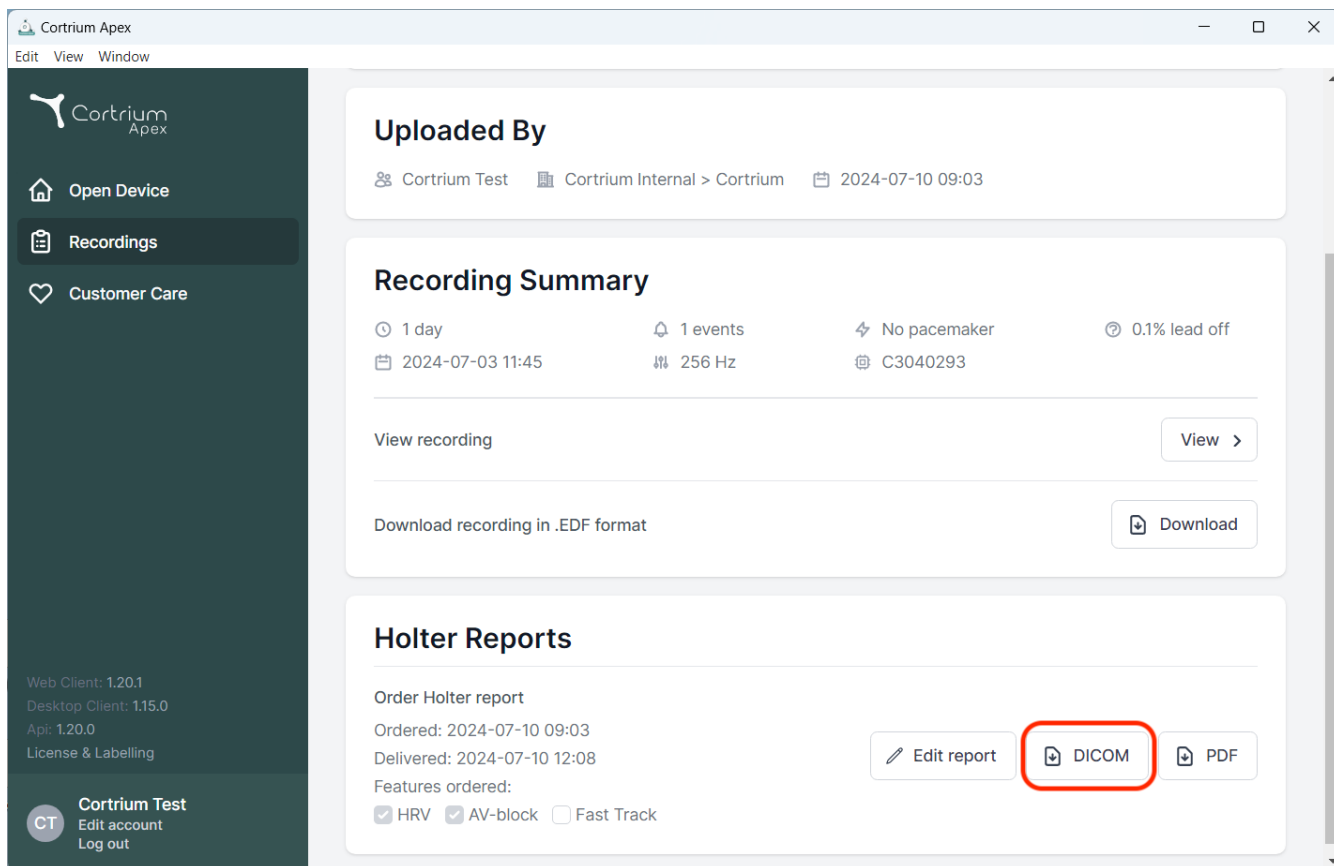
Device setup section:

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

At the bottom of the "Device setup" section, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', 'C3040293', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. The 'DICOM' button is highlighted with a red rectangle.

The bottom left corner shows version information: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. The user 'Cortrium Test' is logged in.

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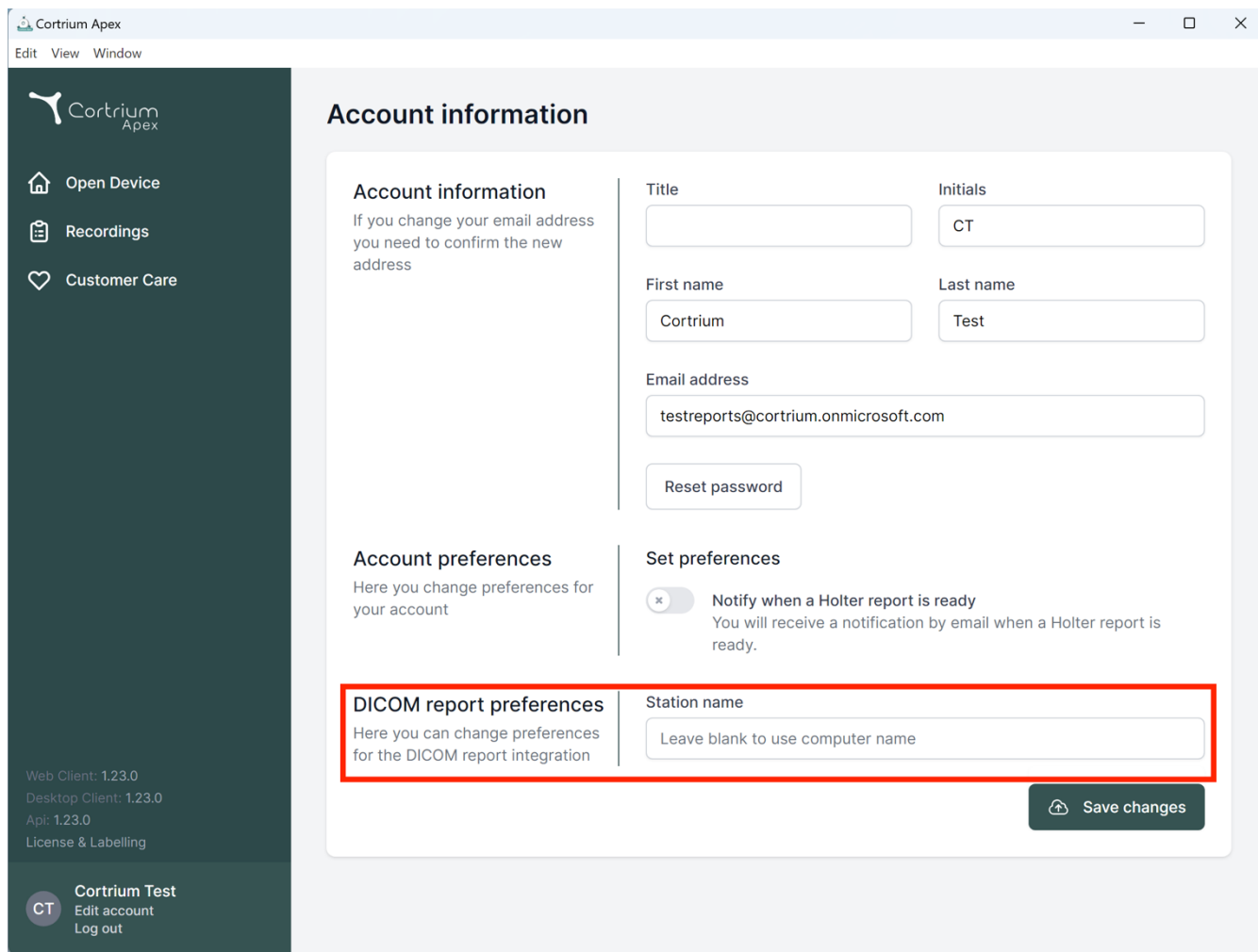Cortrium Apex
Edit View Window

Account information
If you change your email address you need to confirm the new address

Title:
Initials:
First name:
Last name:
Email address:
[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences
☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

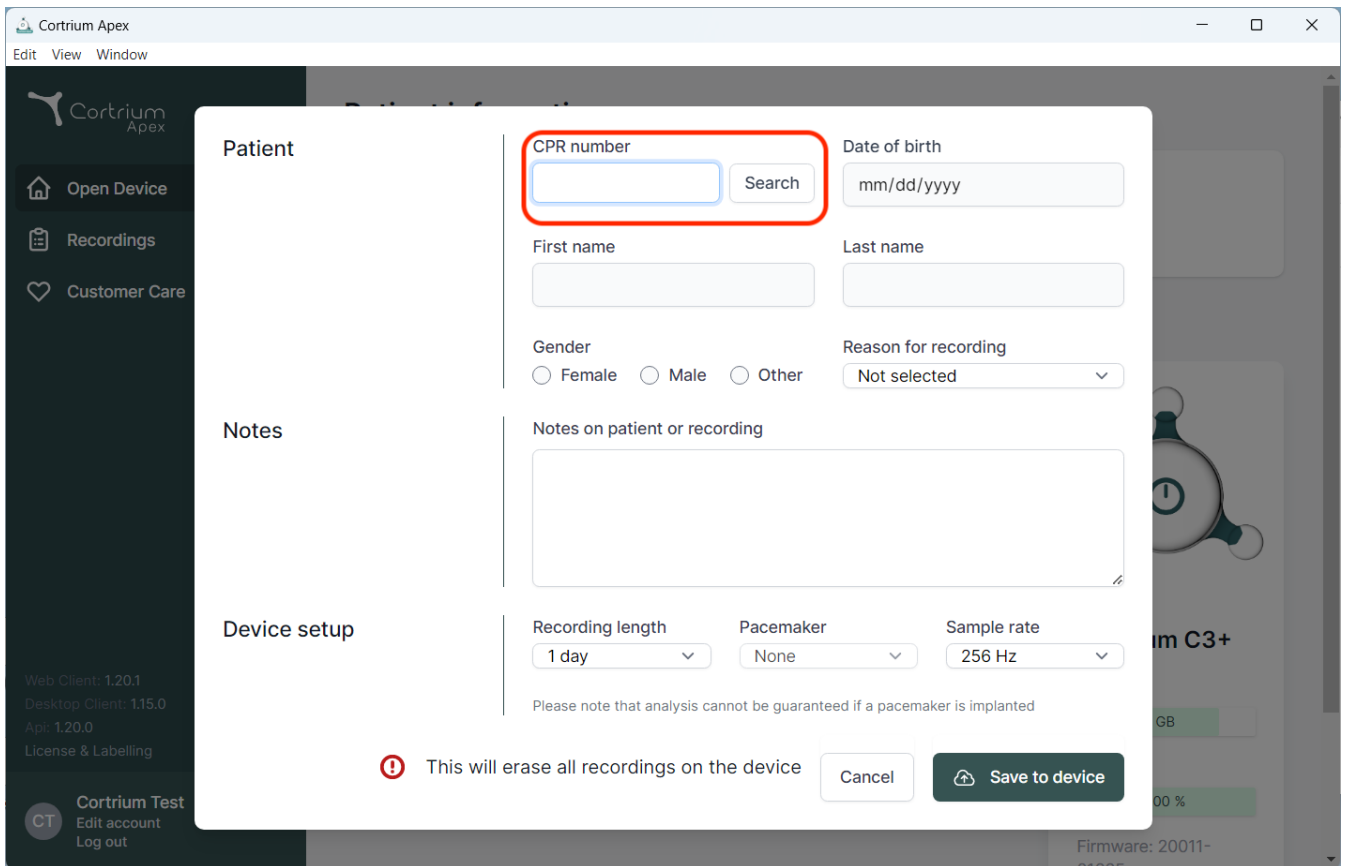
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient search and data entry. The form is divided into several sections:

- Patient:** Contains fields for "CPR number" (highlighted with a red box), "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (radio buttons for Female, Male, Other), and "Reason for recording" (a dropdown menu currently showing "Not selected").
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Includes dropdown menus for "Recording length" (set to 1 day), "Pacemaker" (set to None), and "Sample rate" (set to 256 Hz). Below these is a warning: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the modal, there is a red warning icon and the text "This will erase all recordings on the device". To the right of this text are two buttons: "Cancel" and "Save to device".

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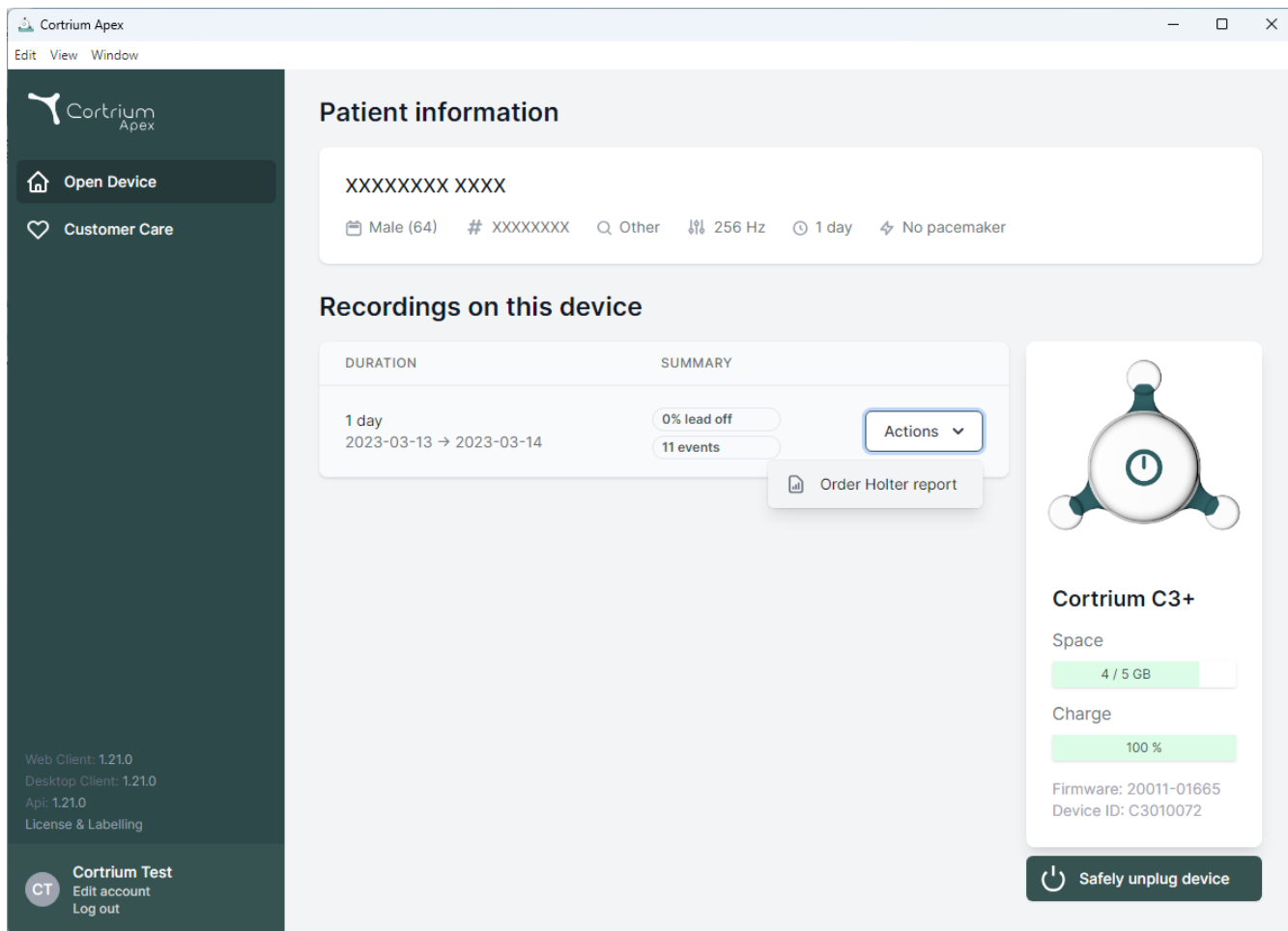
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for a '1 day' recording from '2023-03-13' to '2023-03-14', showing '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right side, a device status card for 'Cortrium C3+' shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of the device card.

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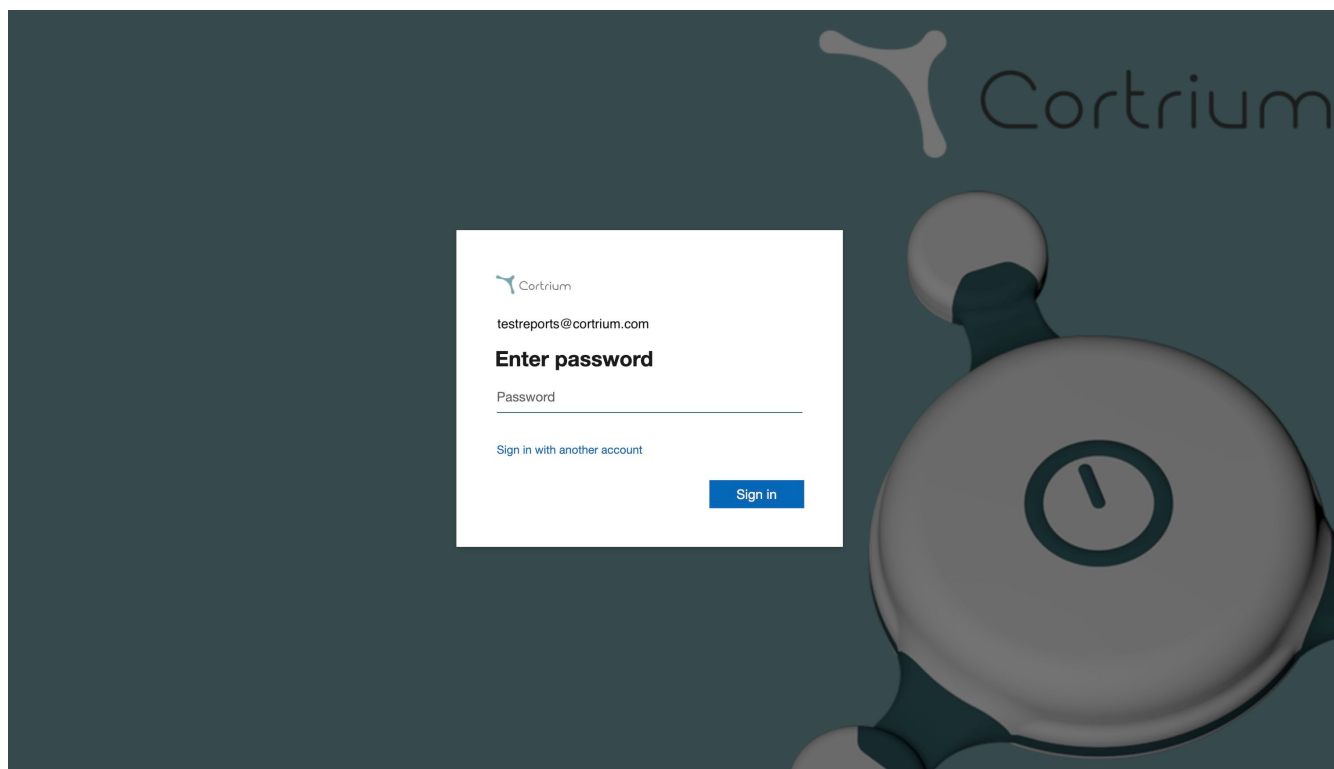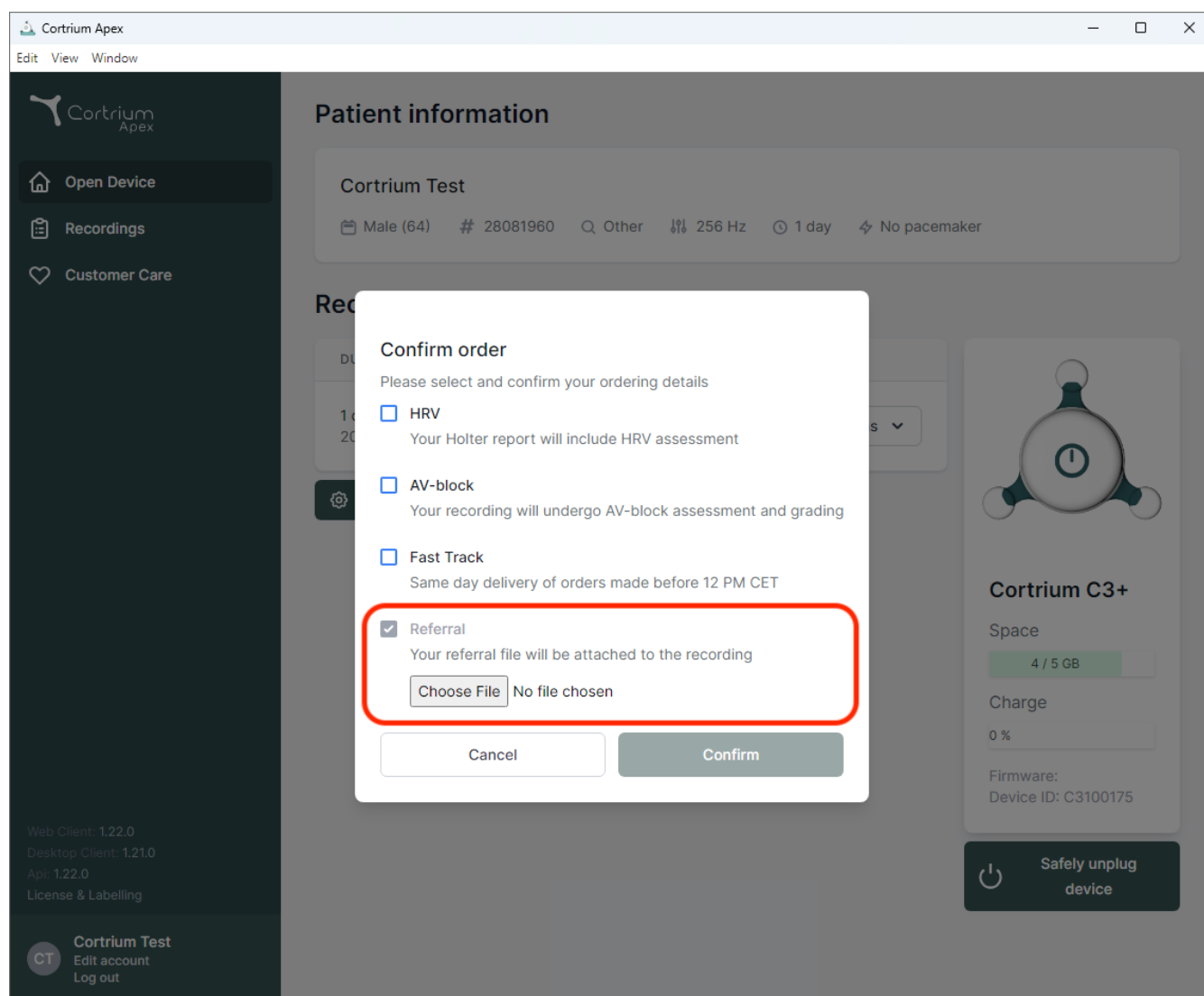
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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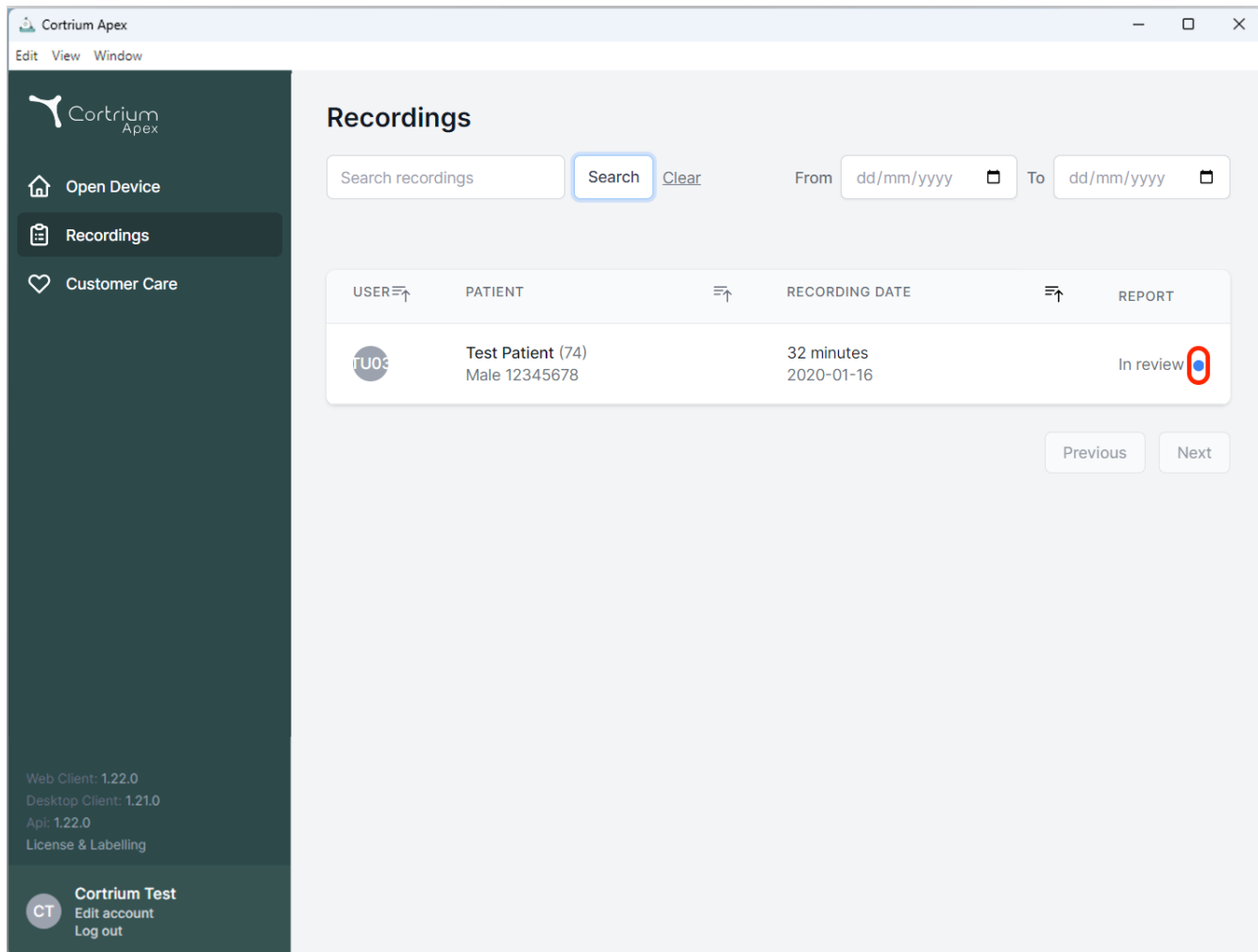
Recordings

Search recordings [Clear](#) From To


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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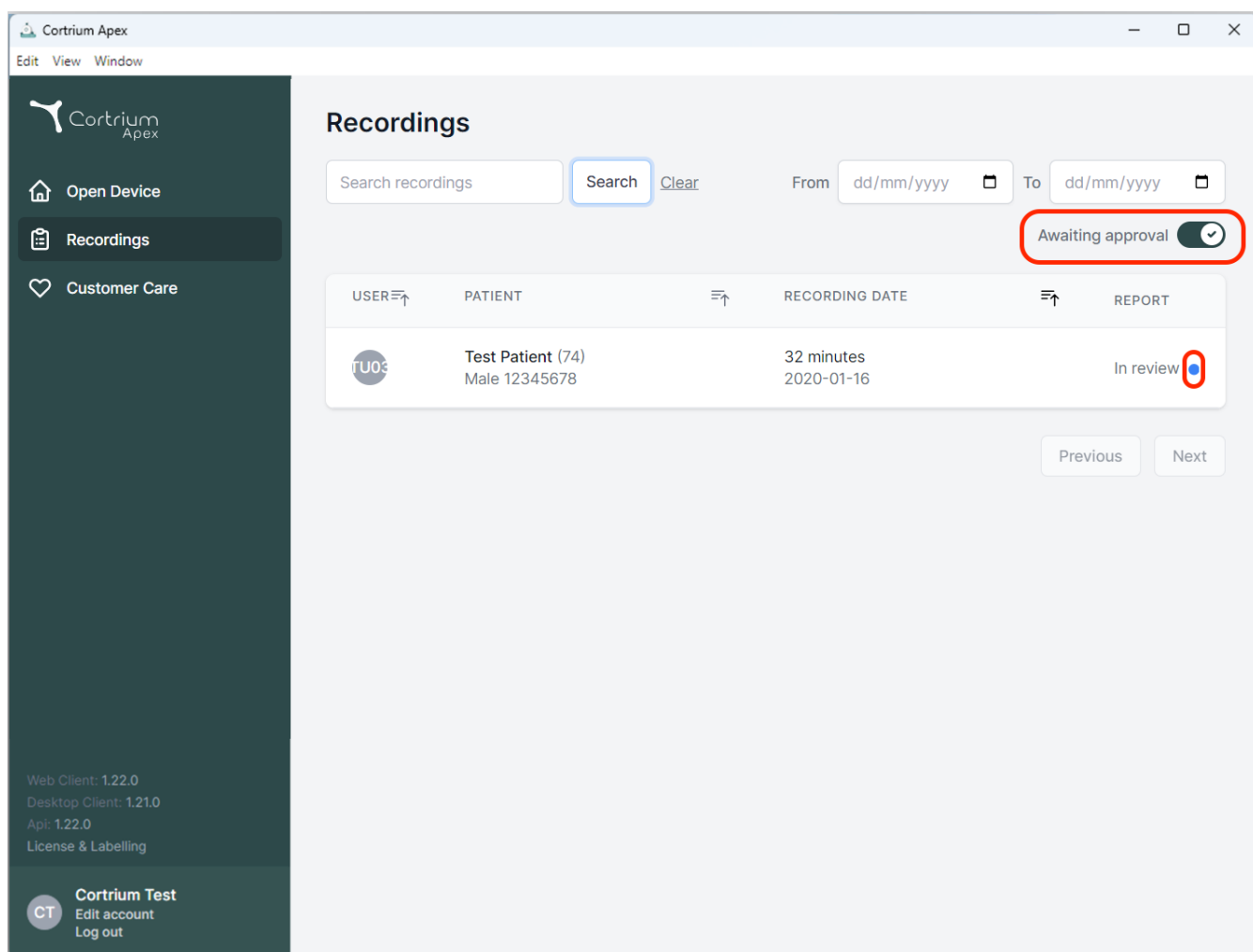Search recordings From To

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●

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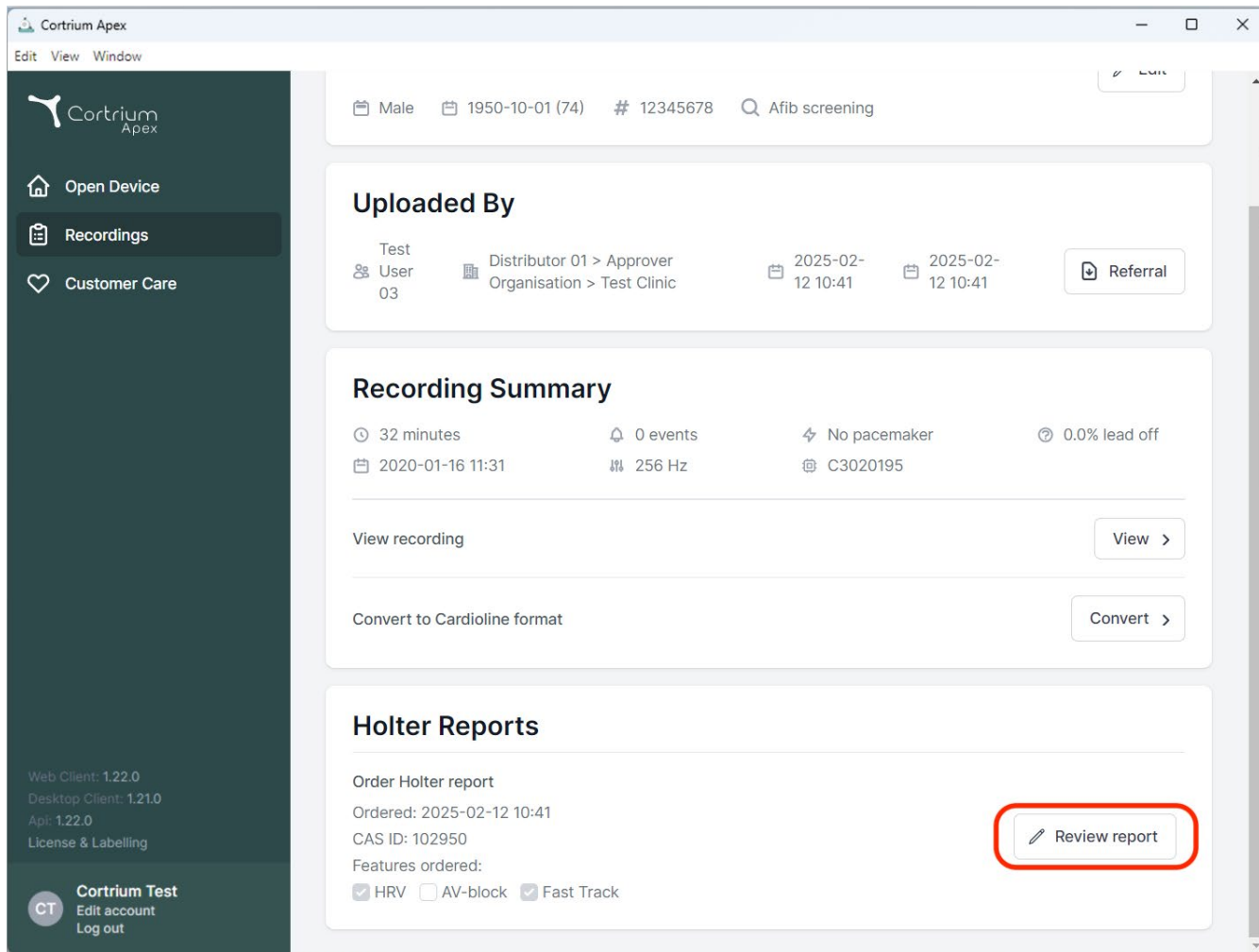
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A red box highlights the 'Review report' button in the bottom right corner of the Holter Reports section.

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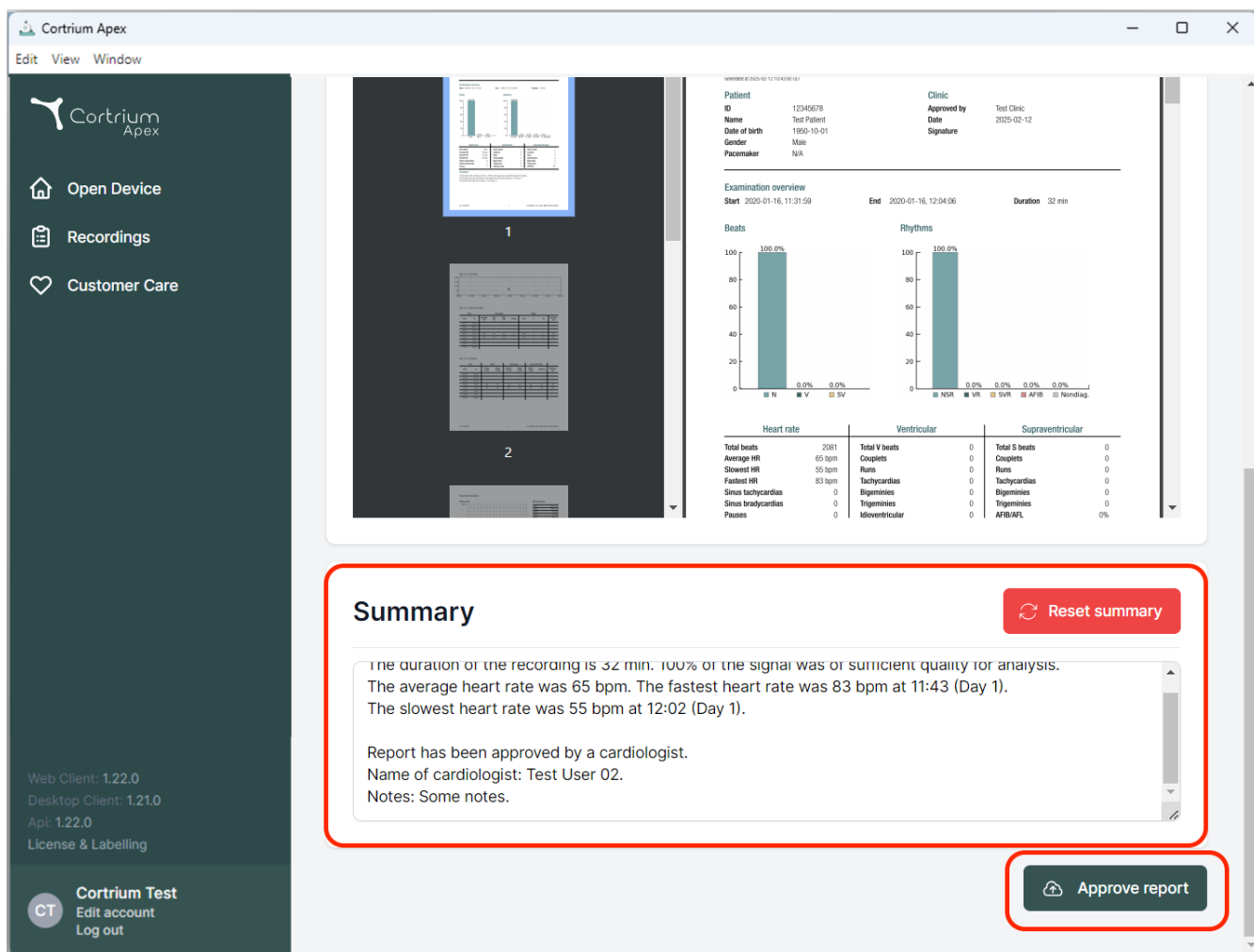The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area shows a patient report for 'Test Patient' (ID: 12345678, DOB: 1990-10-01, Male). The report includes an examination overview (Start: 2020-01-16, End: 2020-01-16, Duration: 32 min) and two bar charts: 'Beats' and 'Rhythms', both showing 100.0% for 'N' (Normal) and 0.0% for other categories. Below these are tables for 'Heart rate', 'Ventricular', and 'Supraventricular' data. A 'Summary' section is highlighted with a red border, containing a 'Reset summary' button and a text area with the following text: 'The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1). Report has been approved by a cardiologist. Name of cardiologist: Test User 02. Notes: Some notes.' At the bottom right, there is an 'Approve report' button, also highlighted with a red border.

Summary Reset summary

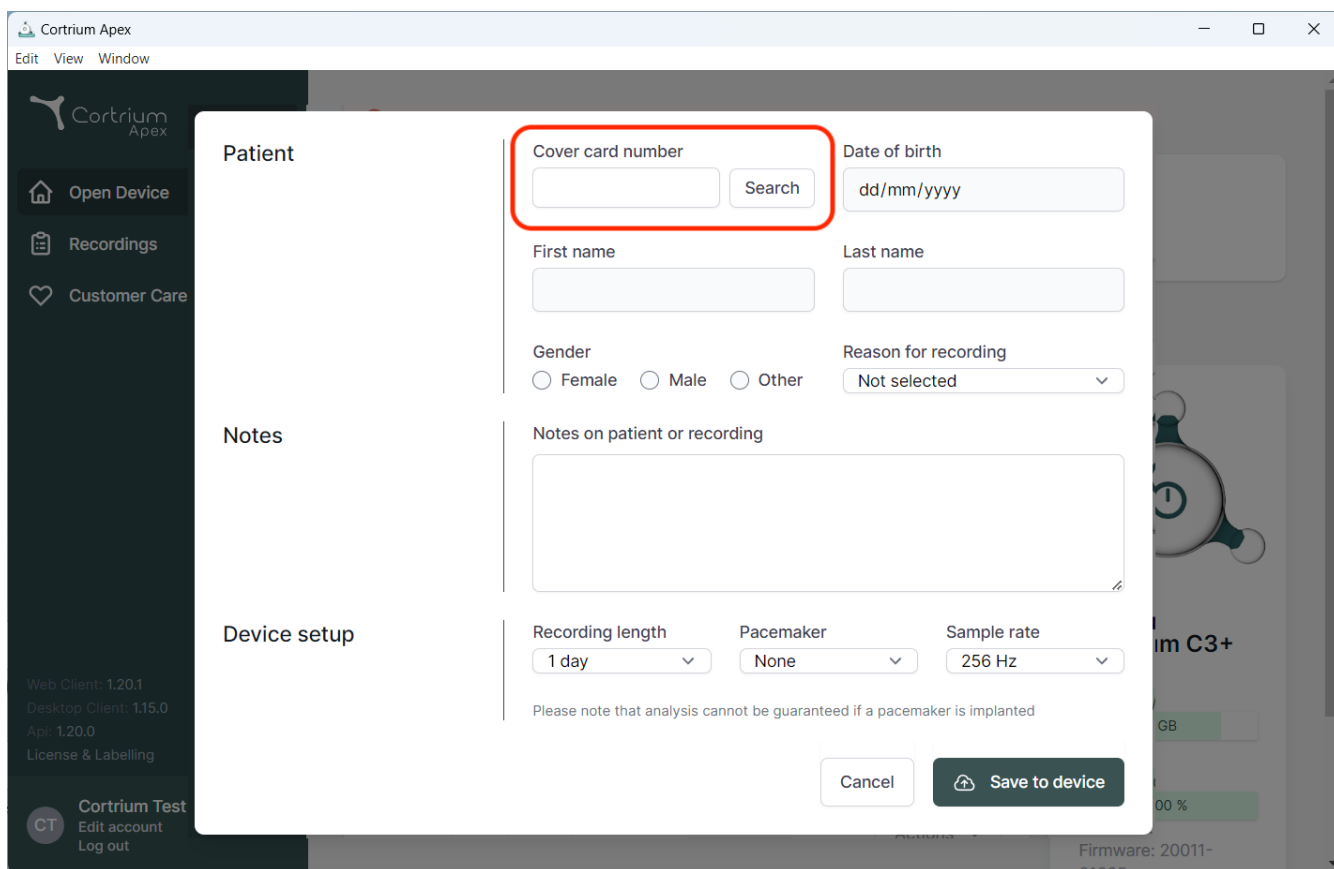
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient information. The "Cover card number" field is highlighted with a red box. The form includes fields for "Date of birth", "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), "Notes on patient or recording", "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A "Cancel" button and a "Save to device" button are at the bottom right. The background shows the Cortrium Apex dashboard with a sidebar menu and a main content area.

Patient

Cover card number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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