Cortrium Apex 1.26

Cortrium Apex 1.26

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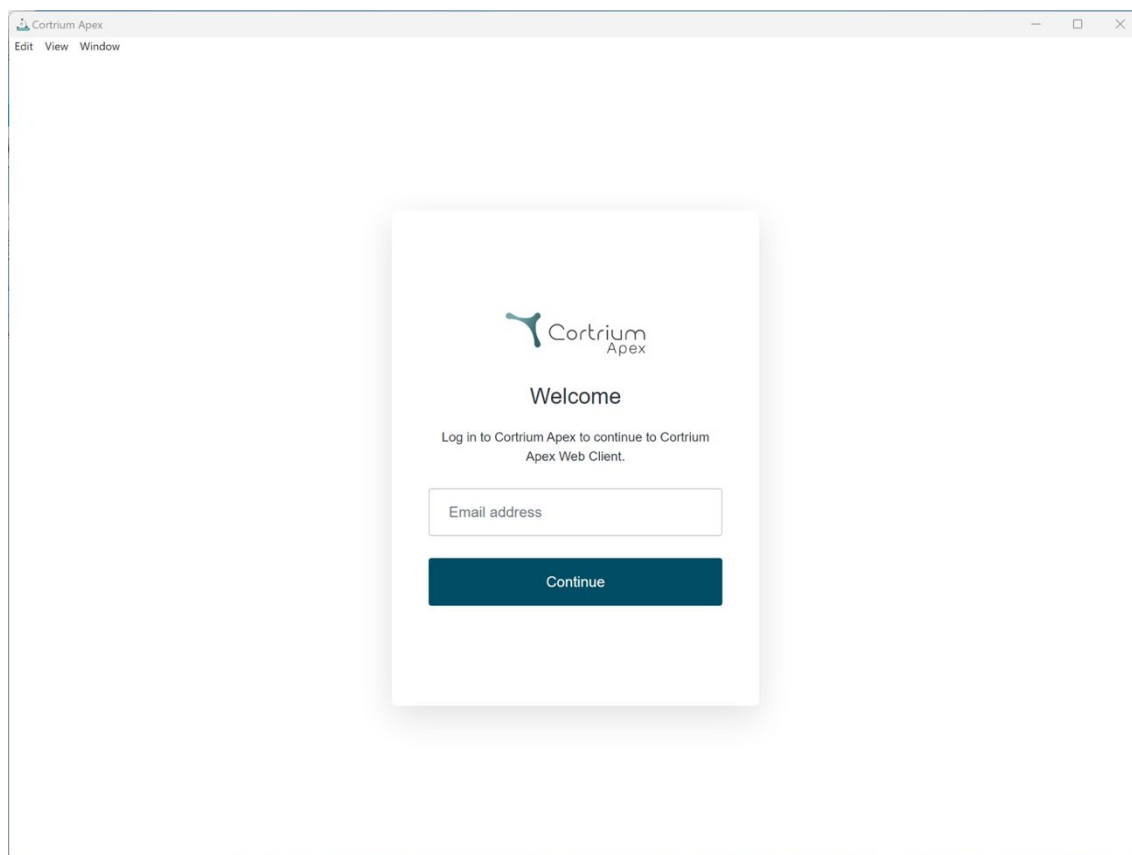
 Cortrium
Apex

Welcome

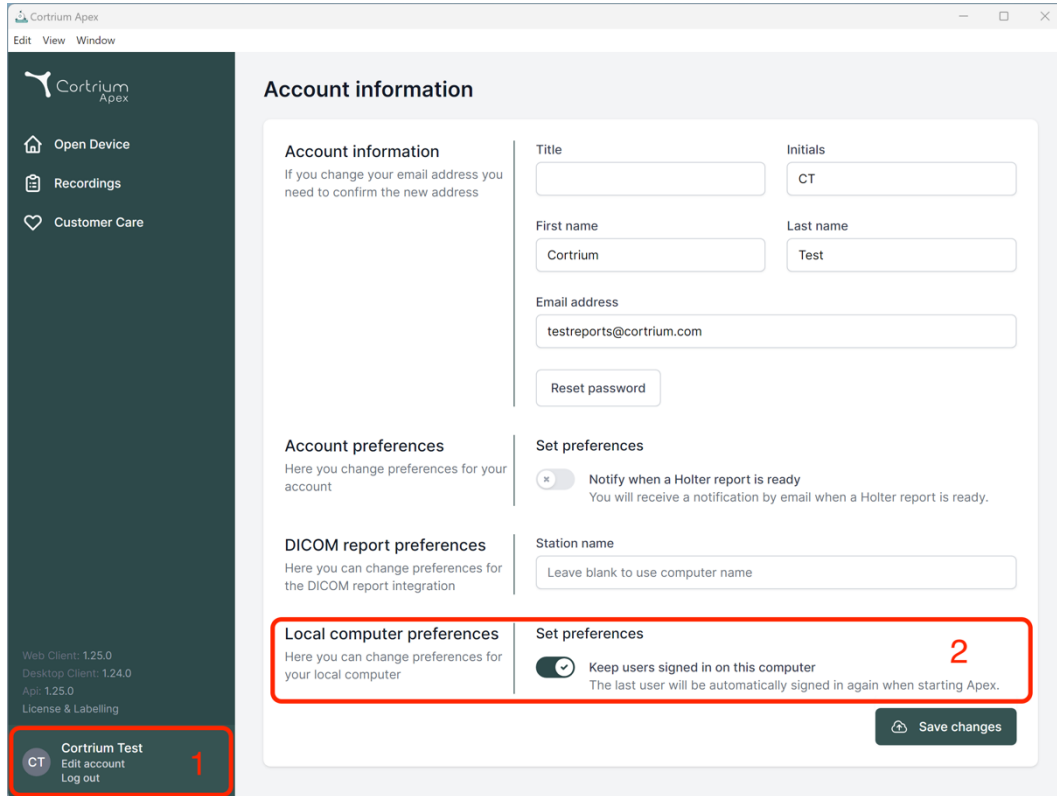
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name

Local computer preferences
Here you can change preferences for your local computer

Set preferences

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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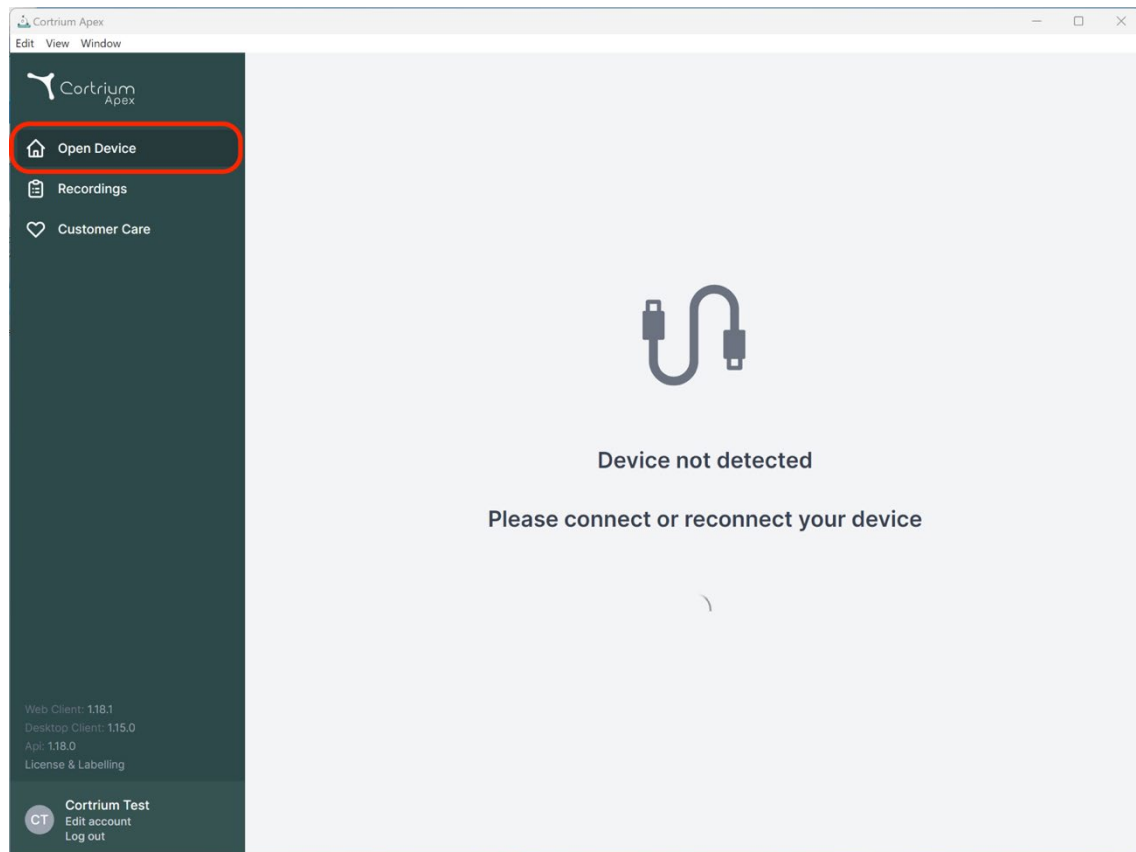
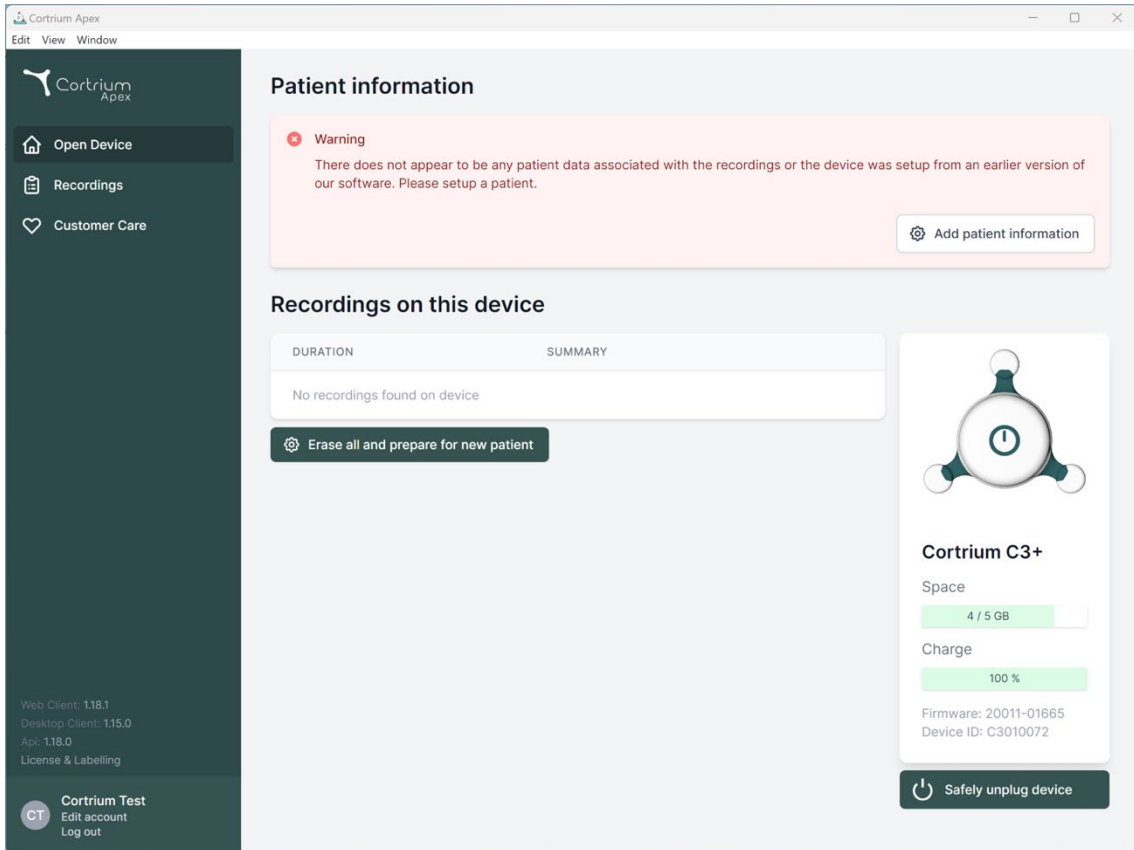
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table header (DURATION, SUMMARY) and a message 'No recordings found on device', accompanied by an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB and 'Charge' at 100 %, along with firmware and device ID information, and a 'Safely unplug device' button.

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Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning This will erase all recordings on the device

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
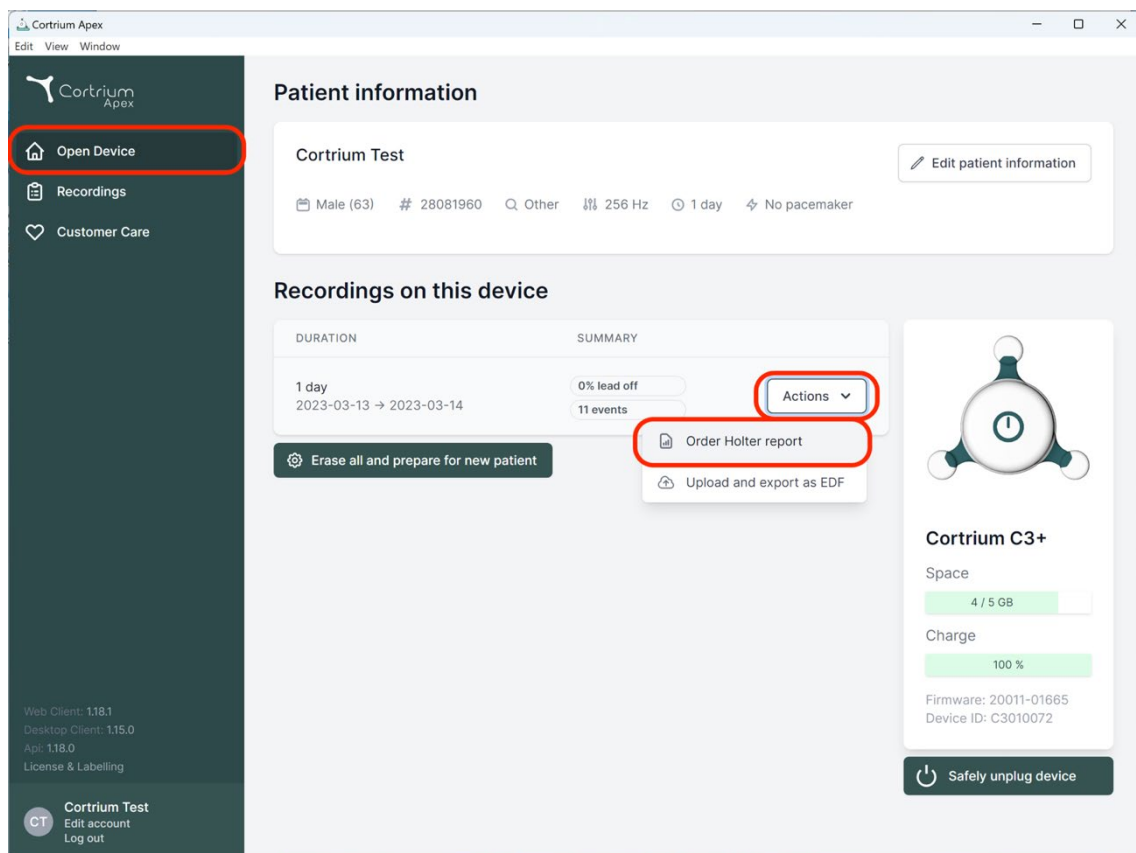
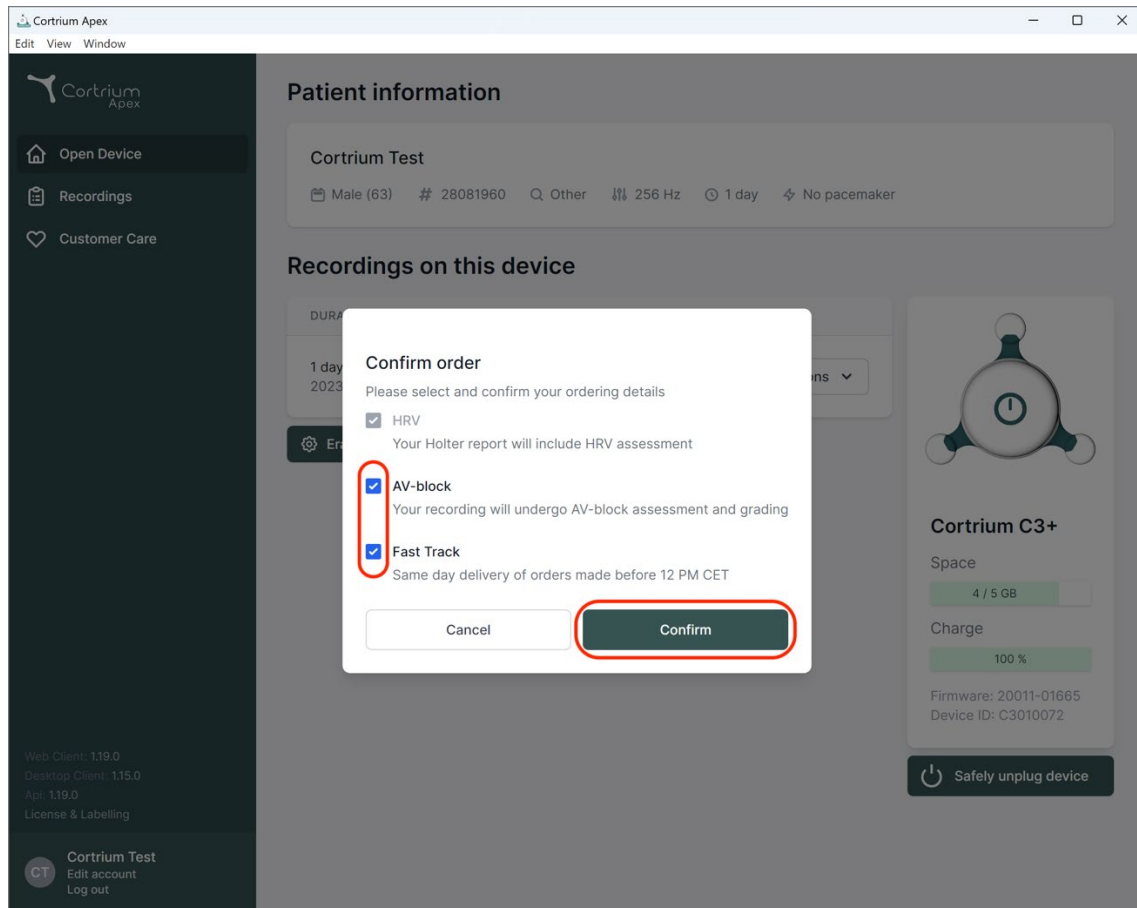
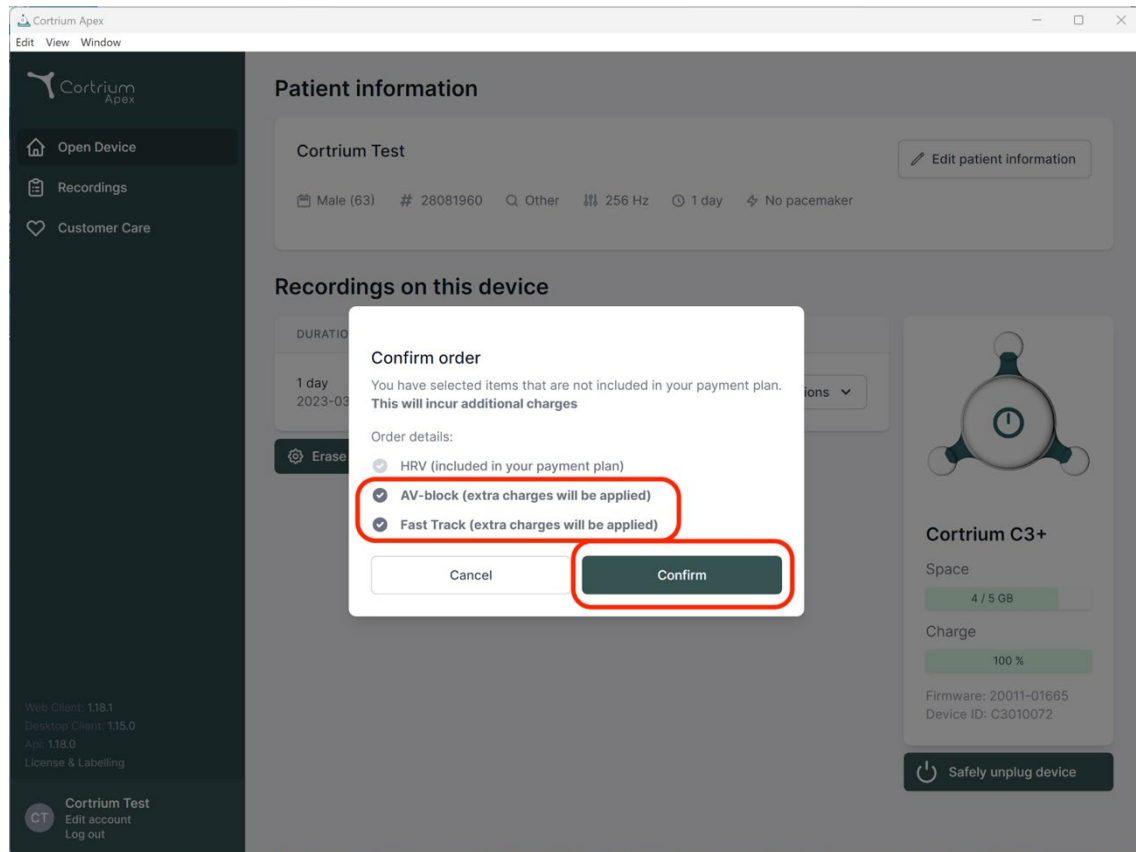
Herefter uploades optagelsen.

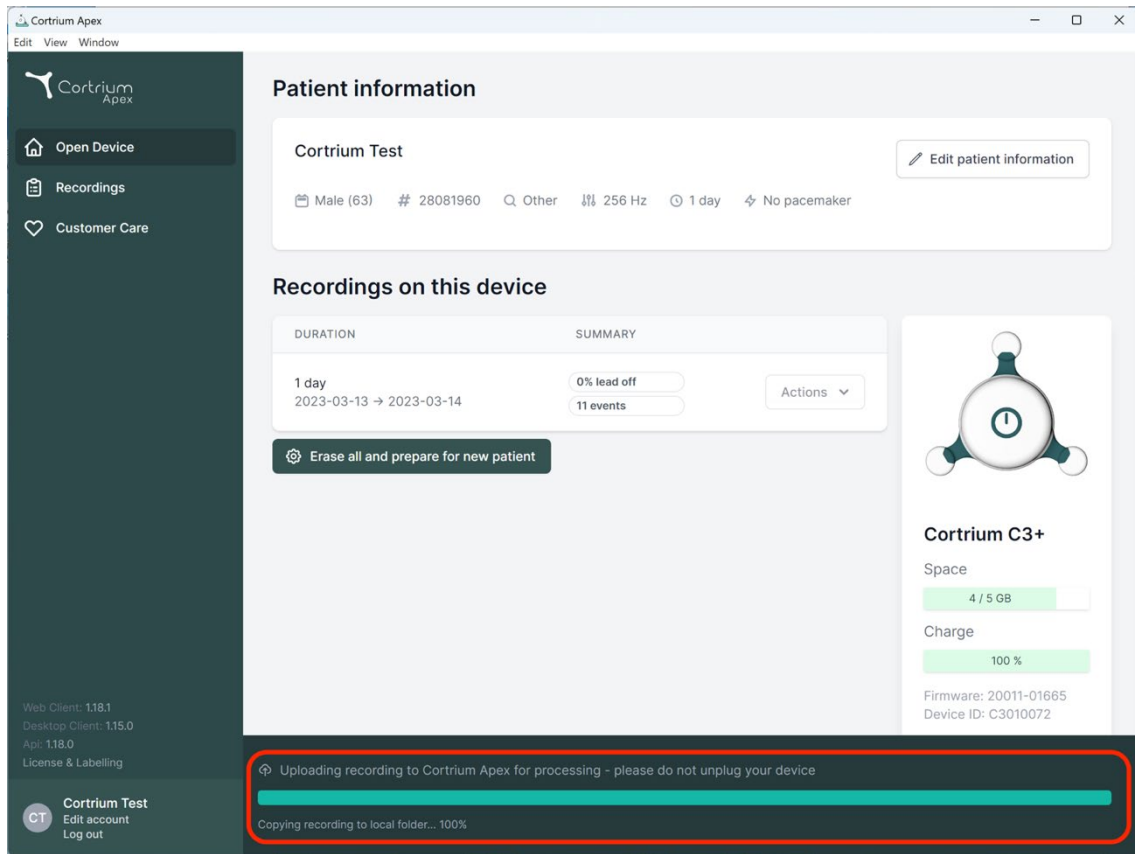The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with an 'Edit account' and 'Log out' option. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and other parameters like '256 Hz' and '1 day'. Below this is a section for 'Recordings on this device' with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. To the right of the table is a device icon and details for 'Cortrium C3+', including space usage (4 / 5 GB), charge (100%), firmware (20011-01665), and device ID (C3010072). At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar for 'Copying recording to local folder... 100%'.

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
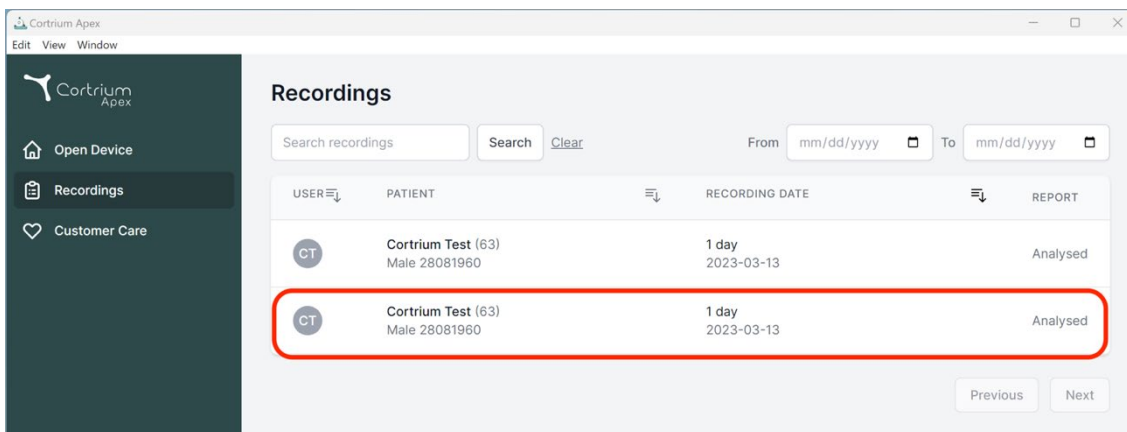
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

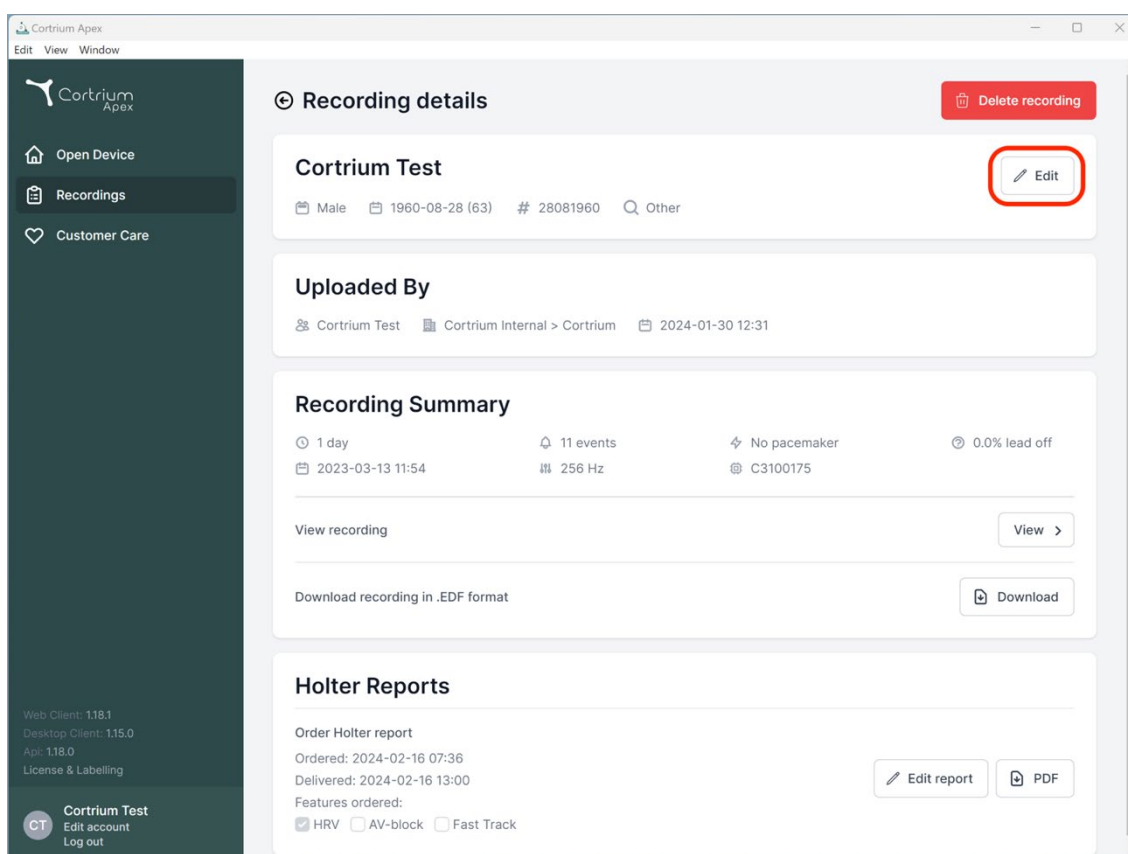Patientoplysningerne kan redigeres ved at klikke på knappen "Rediger".

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
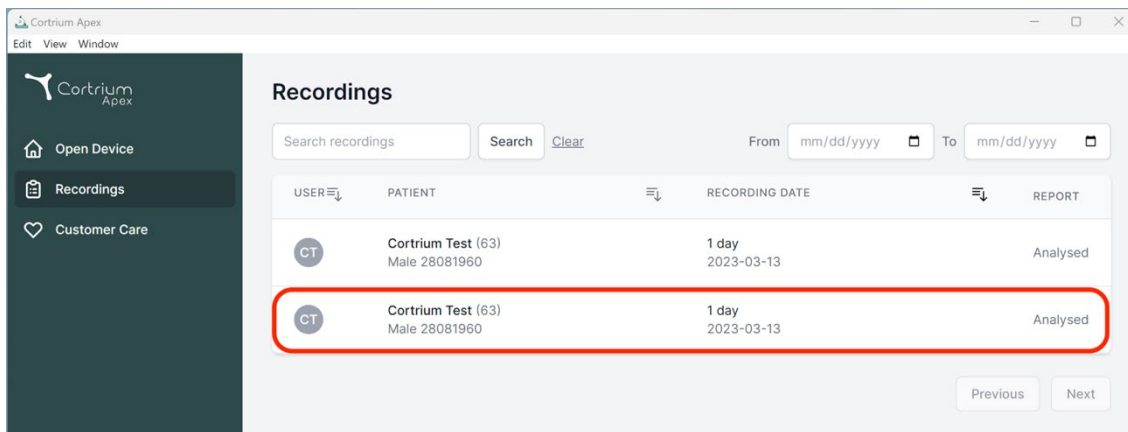
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. A table lists recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red box.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

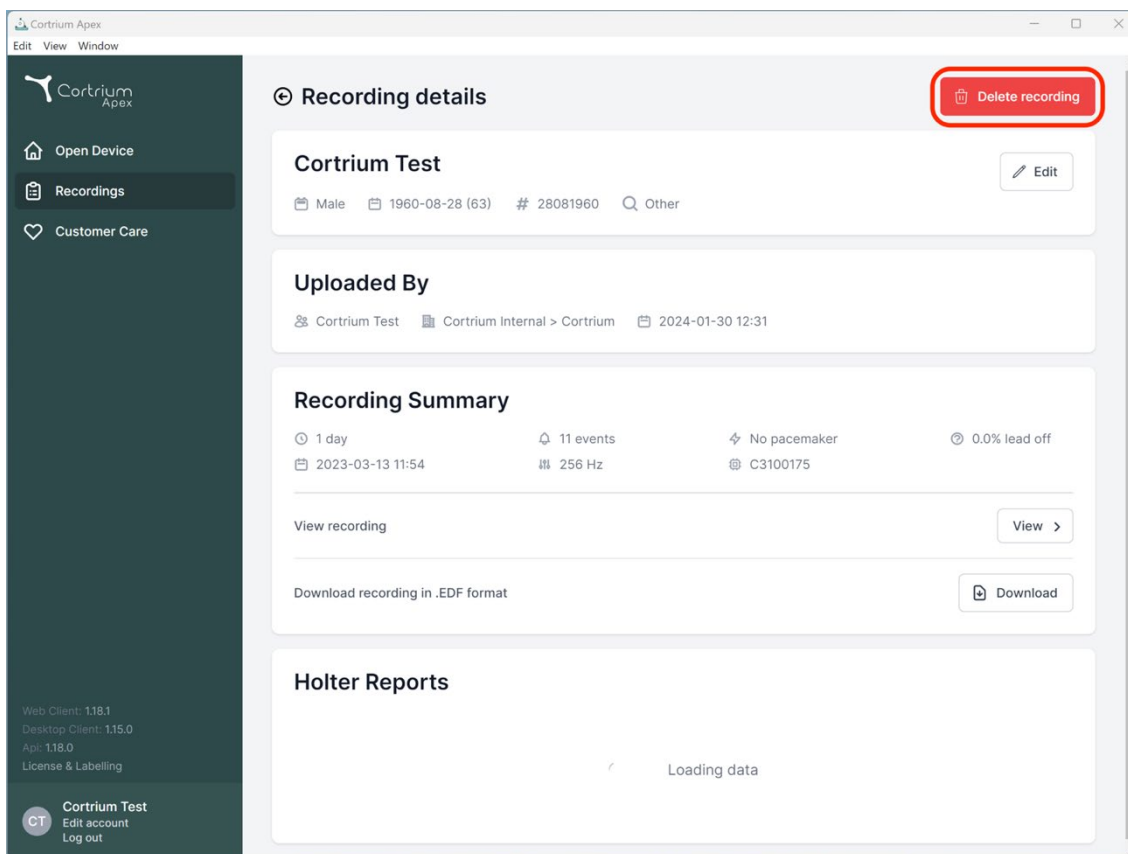The screenshot shows the 'Recording details' page for 'Cortrium Test'. The 'Delete recording' button is highlighted with a red box. The page displays patient information, upload details, recording summary, and holter reports.

Recording details

Cortrium Test [Edit]

Male 1960-08-28 (63) # 28081960 [Other]

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View >]

Download recording in .EDF format [Download]

Holter Reports

Loading data

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

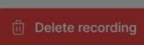
 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

 Delete recording

Cortrium Test 

 Male  1960-08-28 (63)  # 28081960  Other

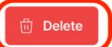
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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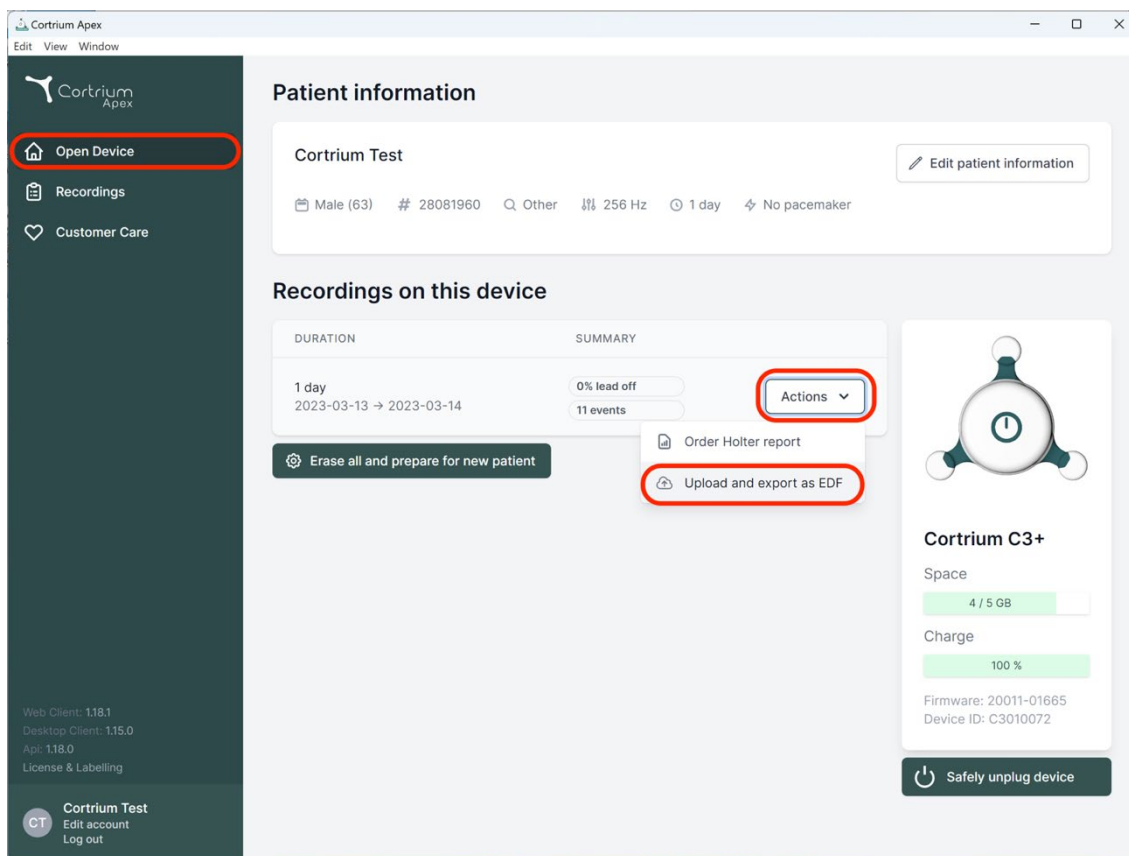
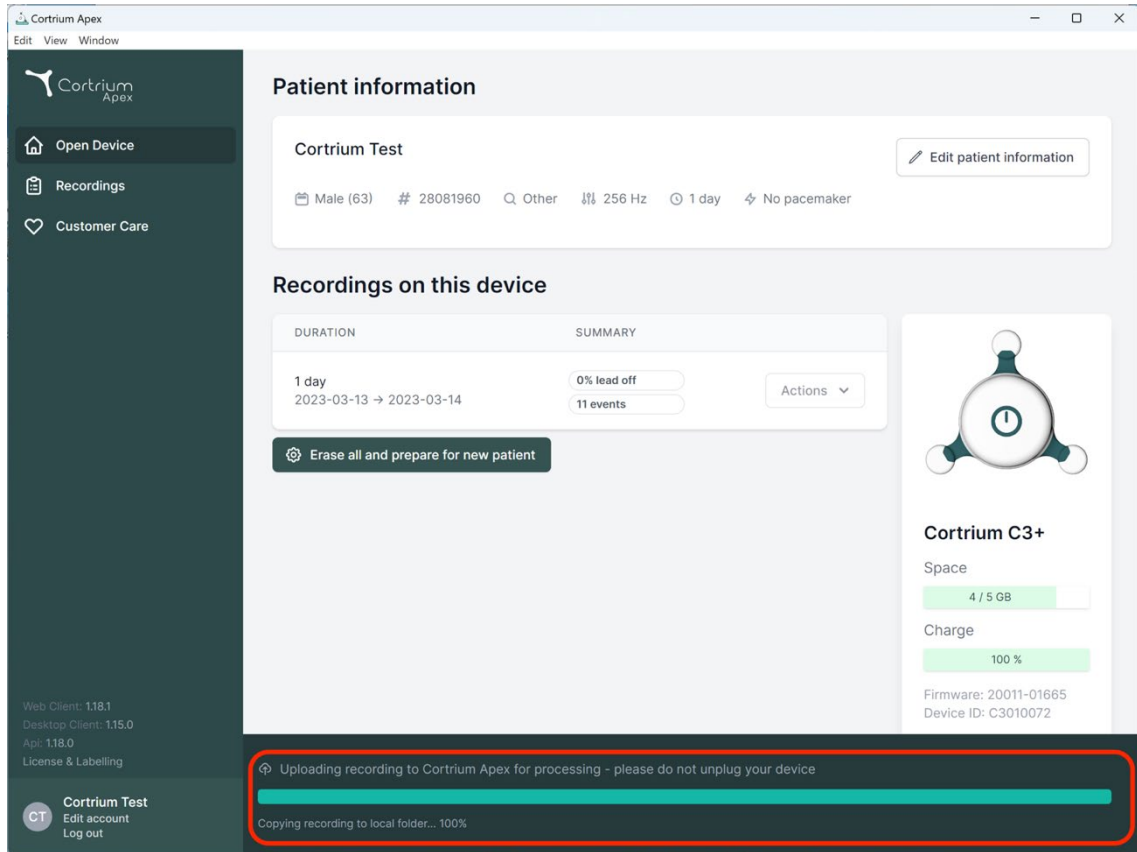
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

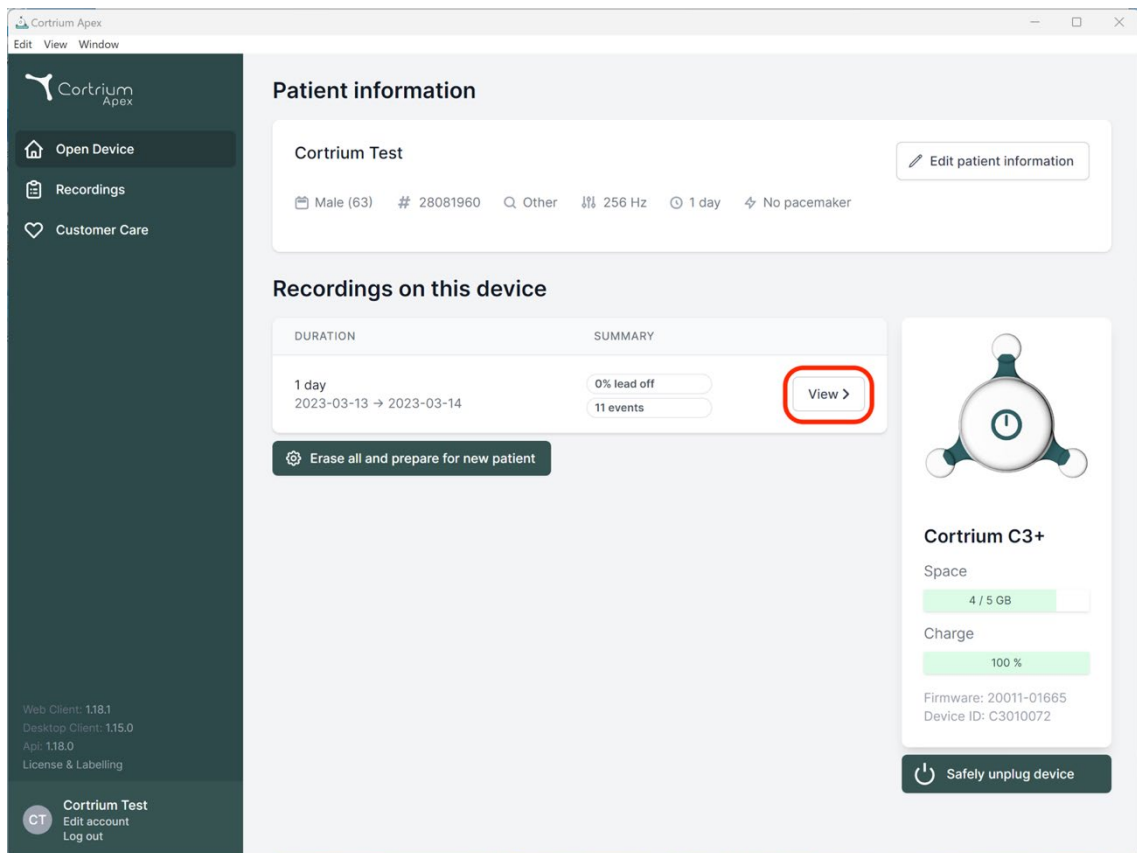
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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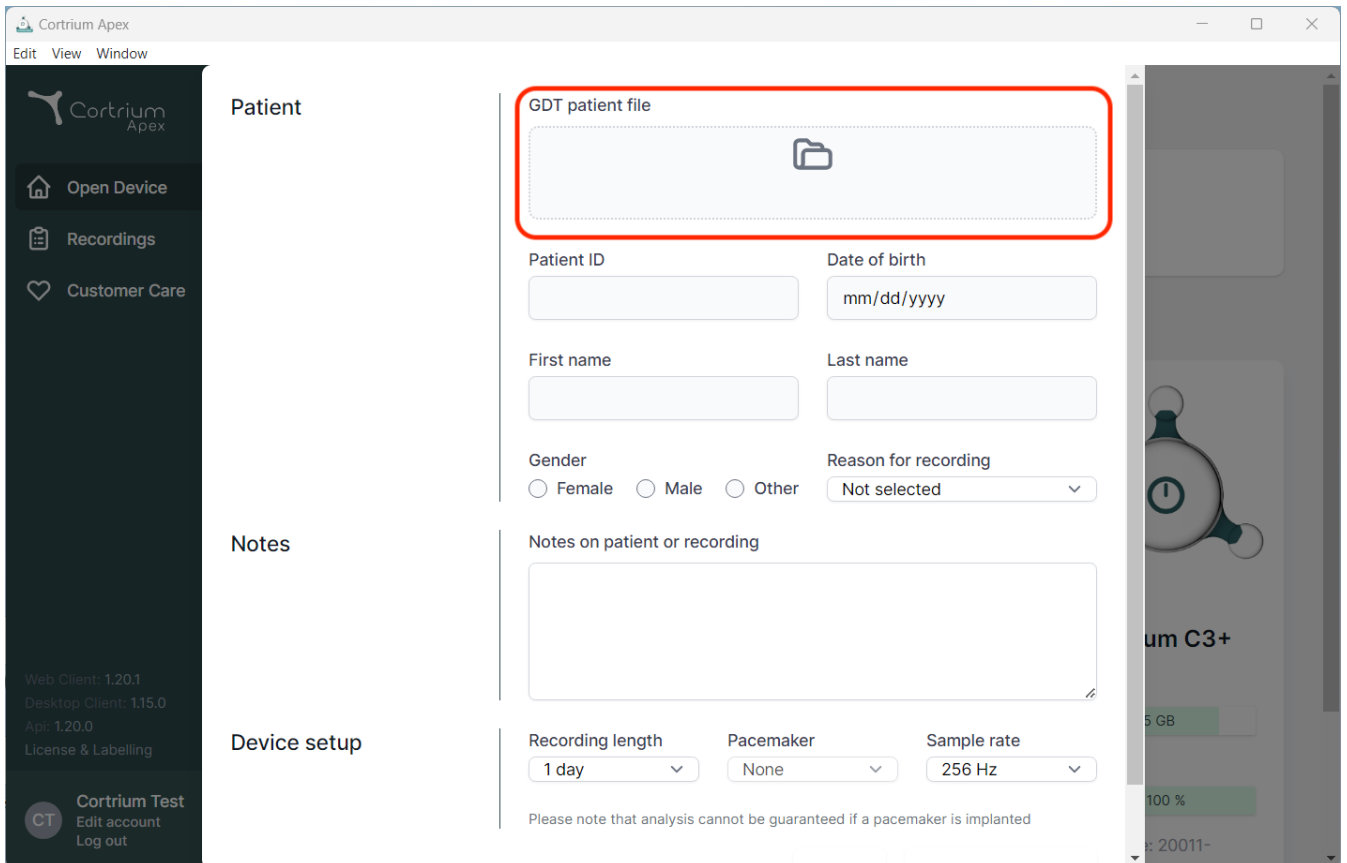
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The GDT patient file field is highlighted with a red box. Below it are fields for Patient ID, Date of birth, First name, Last name, Gender, and Reason for recording. The Notes section contains a text area for notes on the patient or recording. The Device setup section contains dropdown menus for Recording length, Pacemaker, and Sample rate. A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

Patient

GDT patient file

Patient ID

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

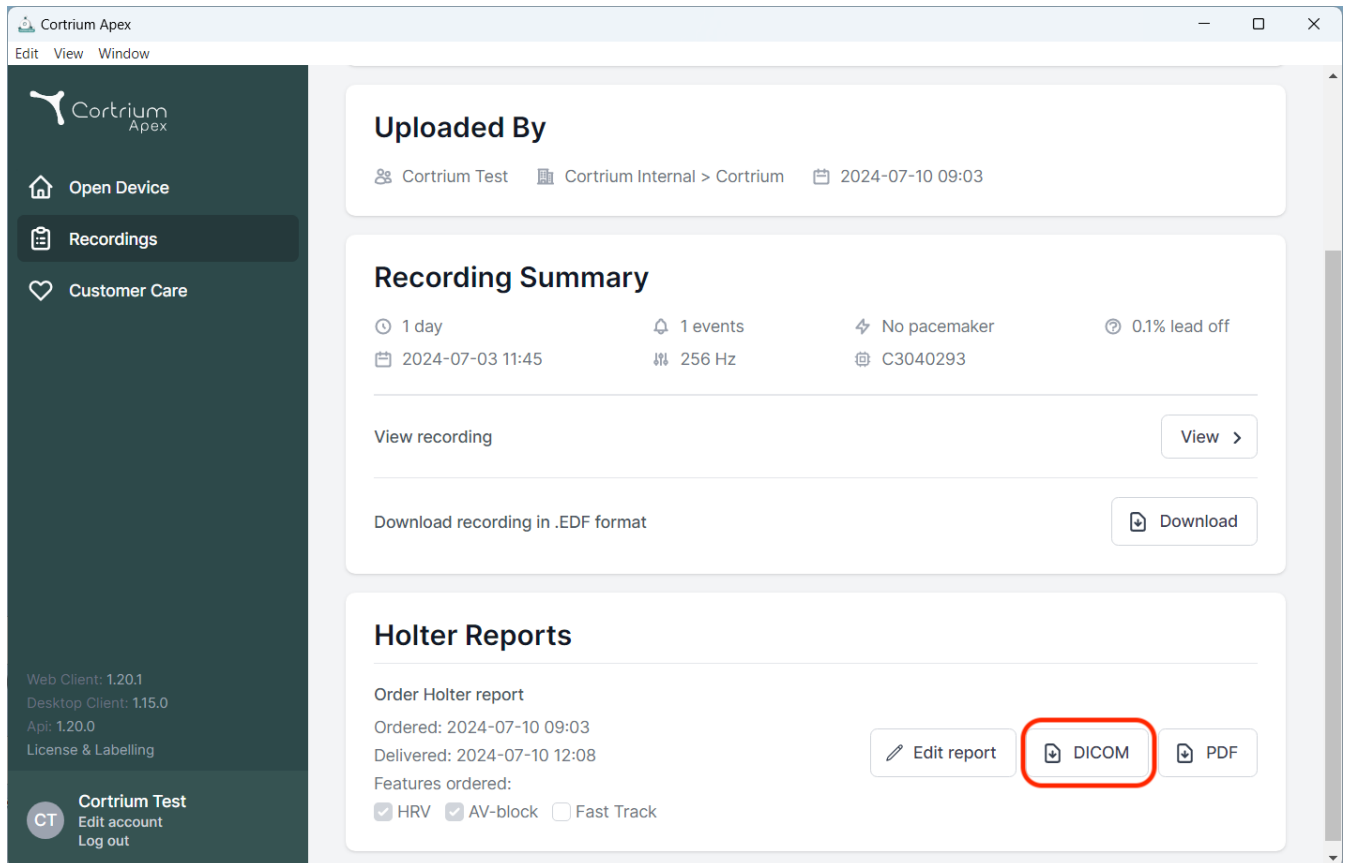
Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

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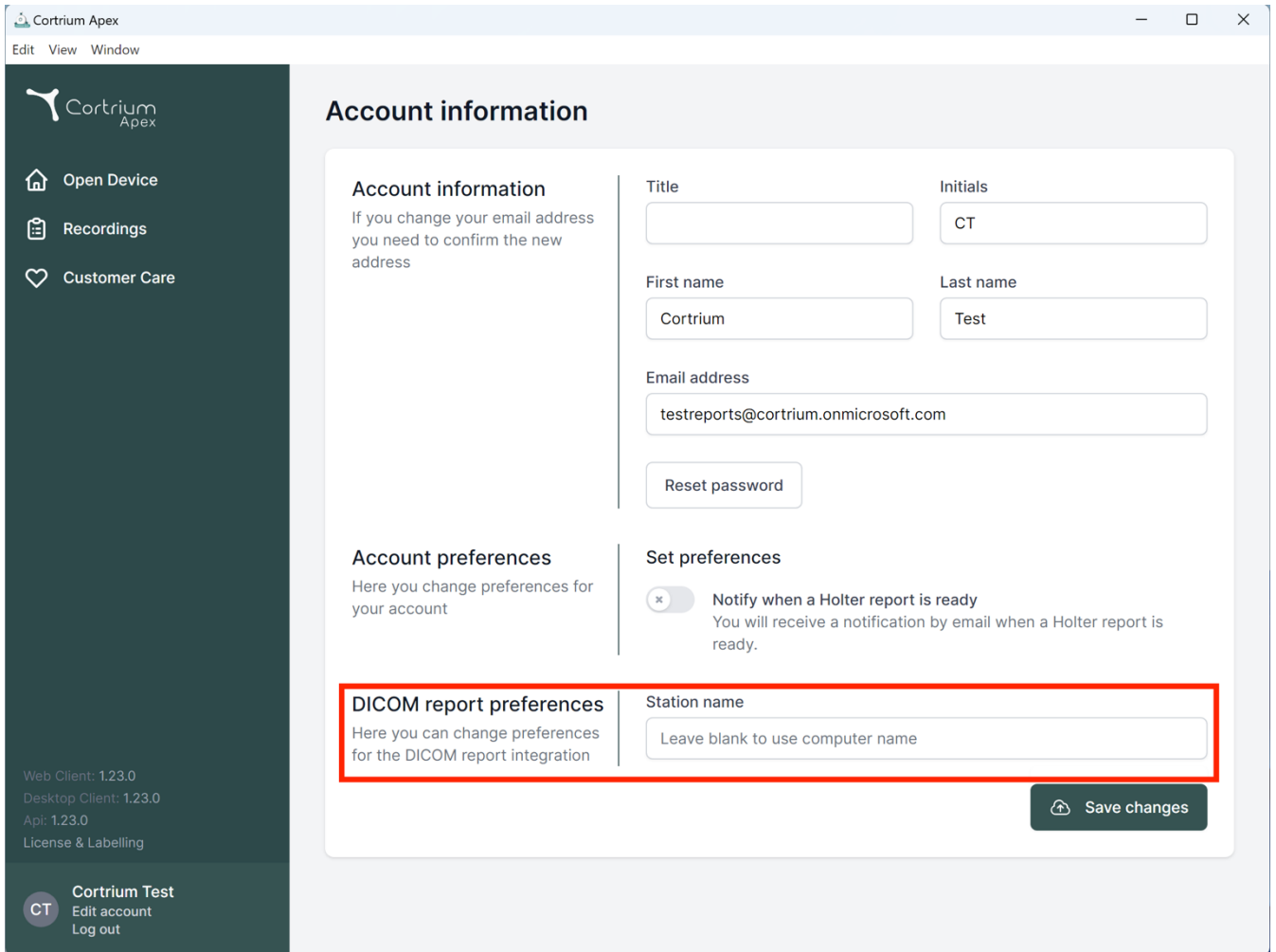The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' with a note about email changes, 'Account preferences' for general account settings, 'Set preferences' for notifications, and 'DICOM report preferences' for integration settings. The 'DICOM report preferences' section is highlighted with a red box and contains a 'Station name' field with the placeholder text 'Leave blank to use computer name'. A 'Save changes' button is located at the bottom right of the form.

Account information

If you change your email address you need to confirm the new address

Account preferences

Here you change preferences for your account

Set preferences

☐ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

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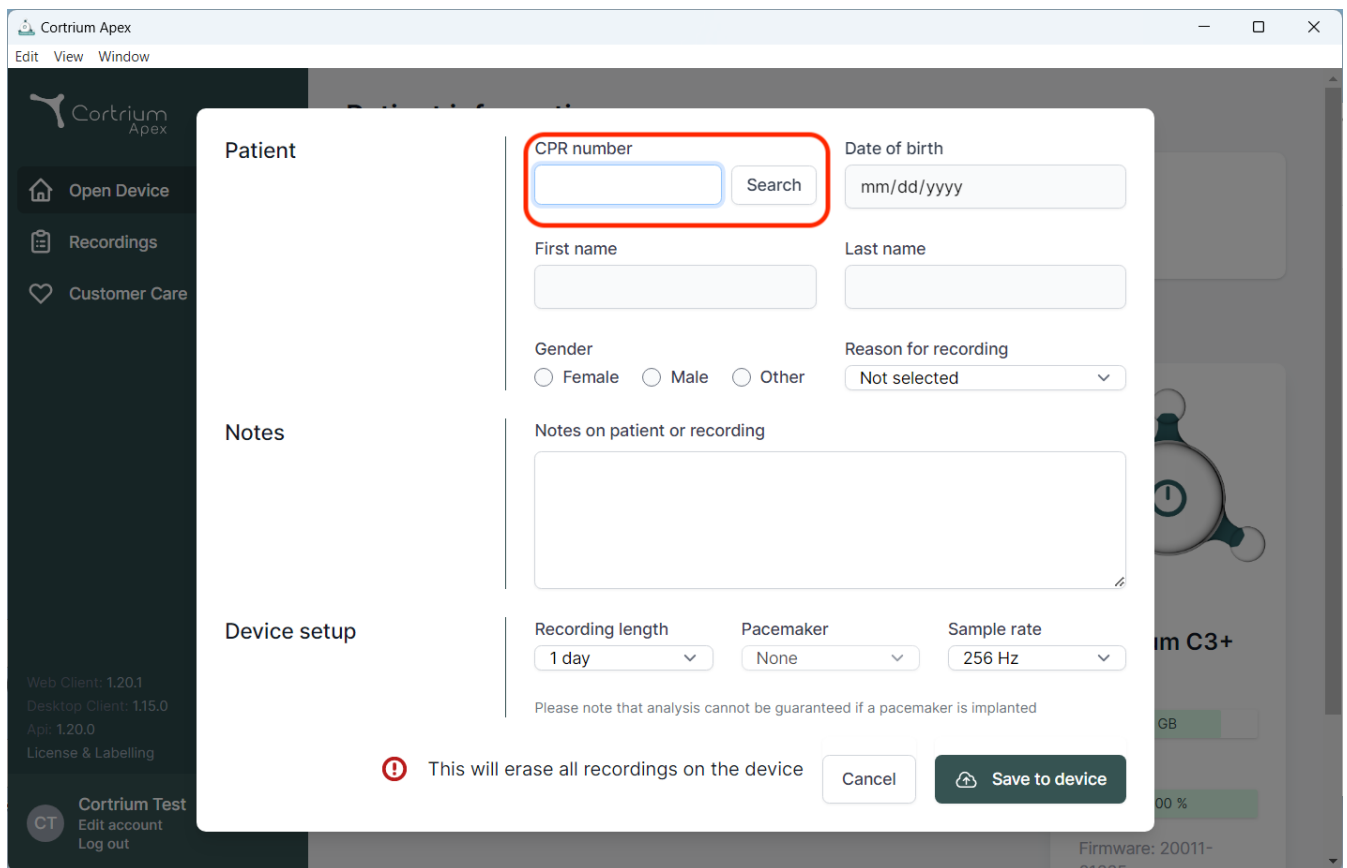
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient information. The form is divided into sections: "Patient", "Notes", and "Device setup".

Patient section:

- CPR number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup section:

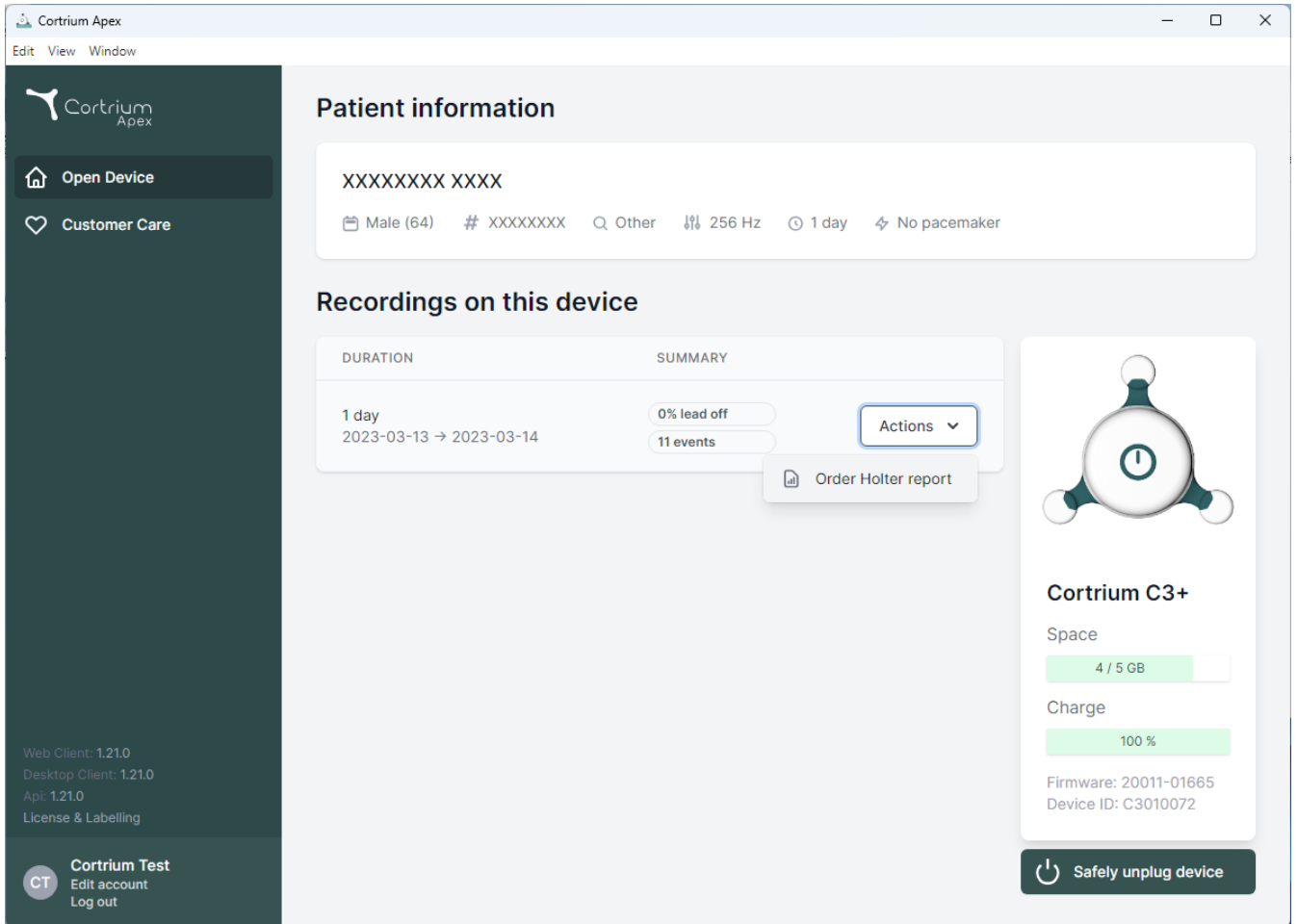
- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the modal, there is a red warning icon and the text: "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Order Holter report

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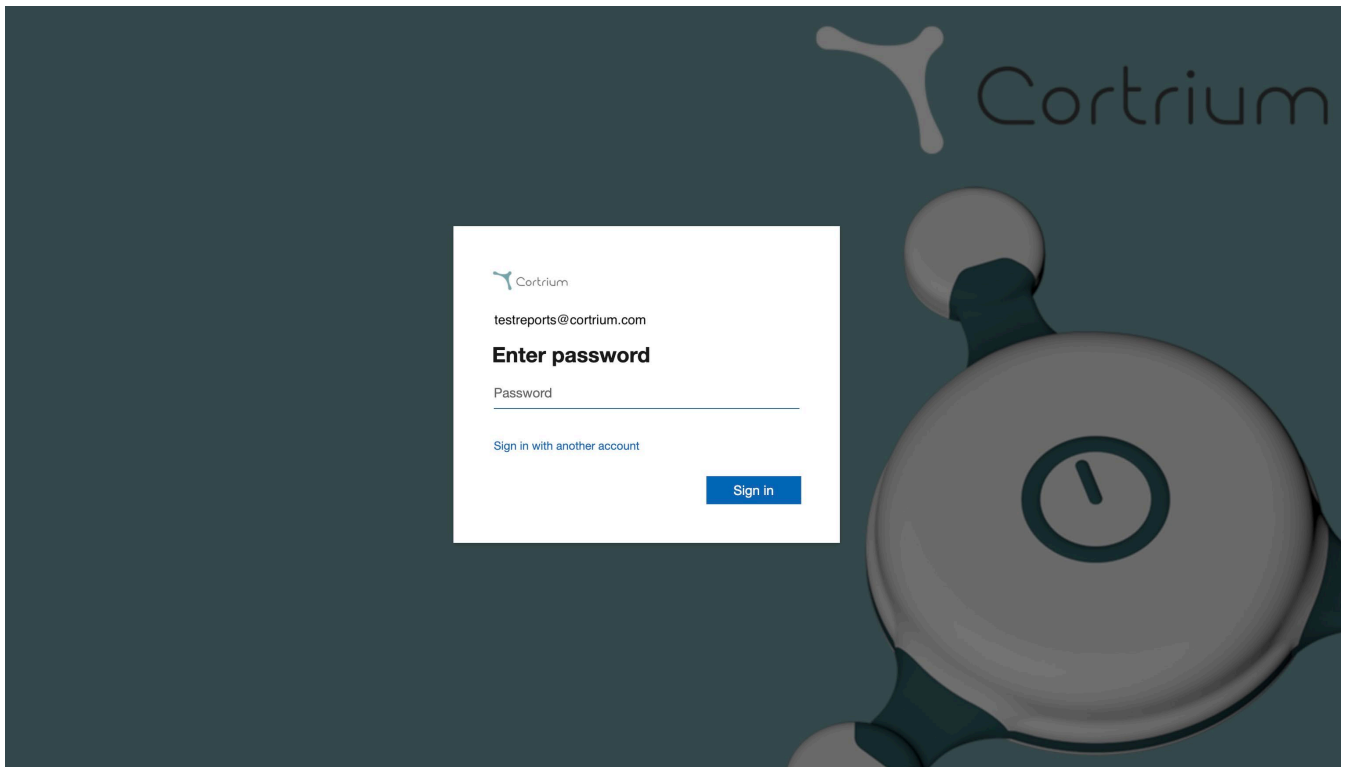
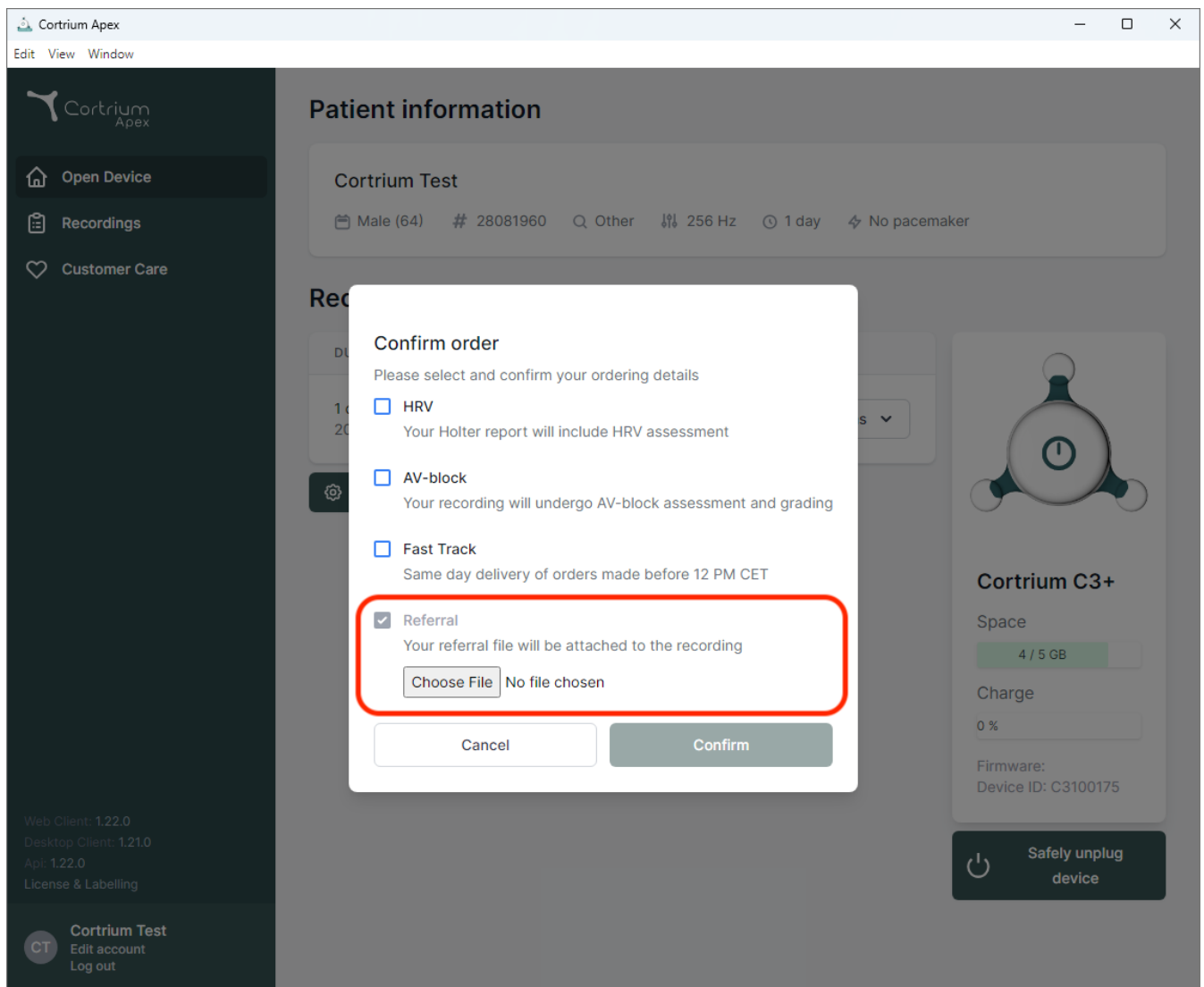
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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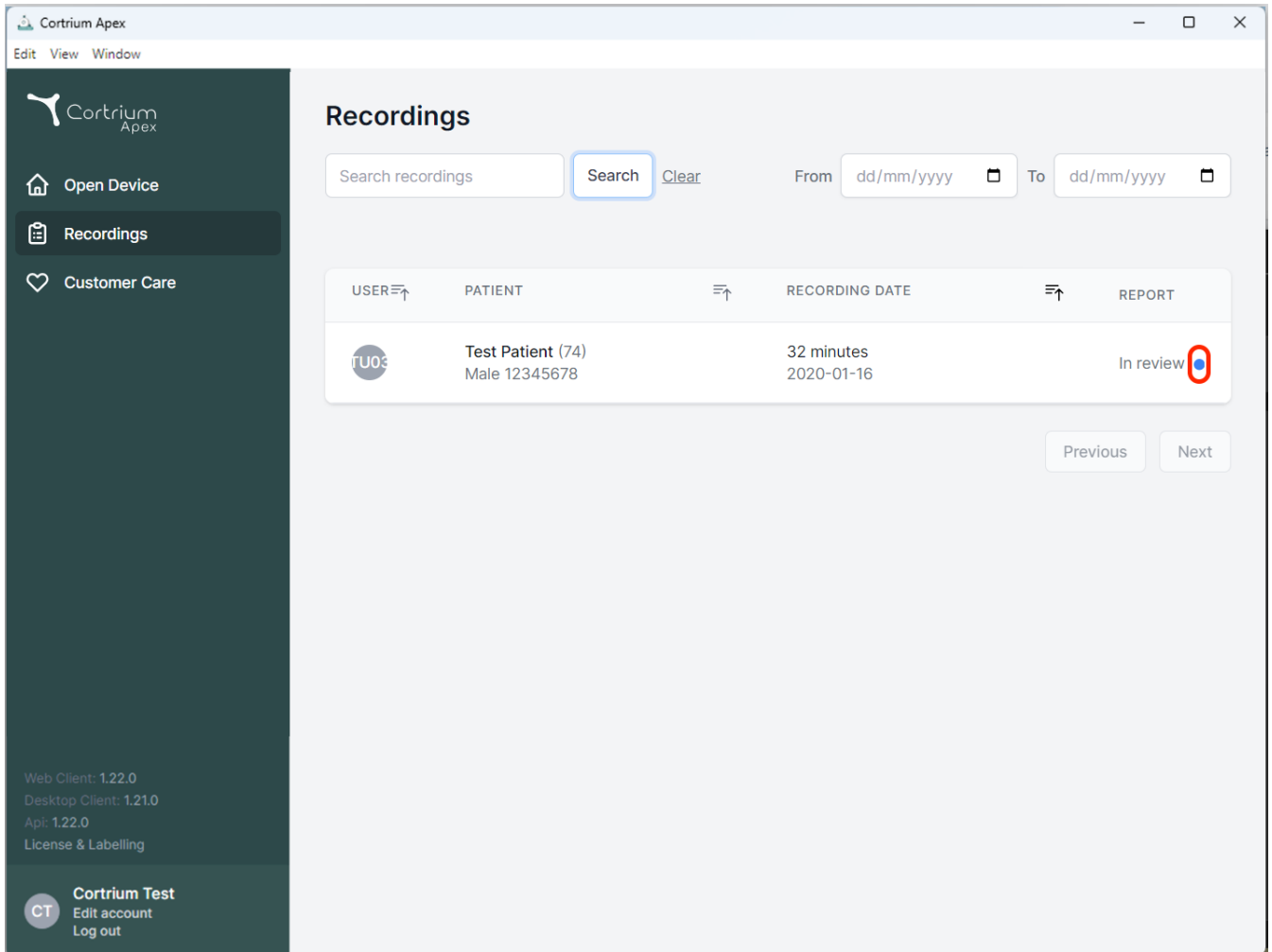
Edit View Window

Recordings

Search recordings Search Clear From dd/mm/yyyy To dd/mm/yyyy

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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
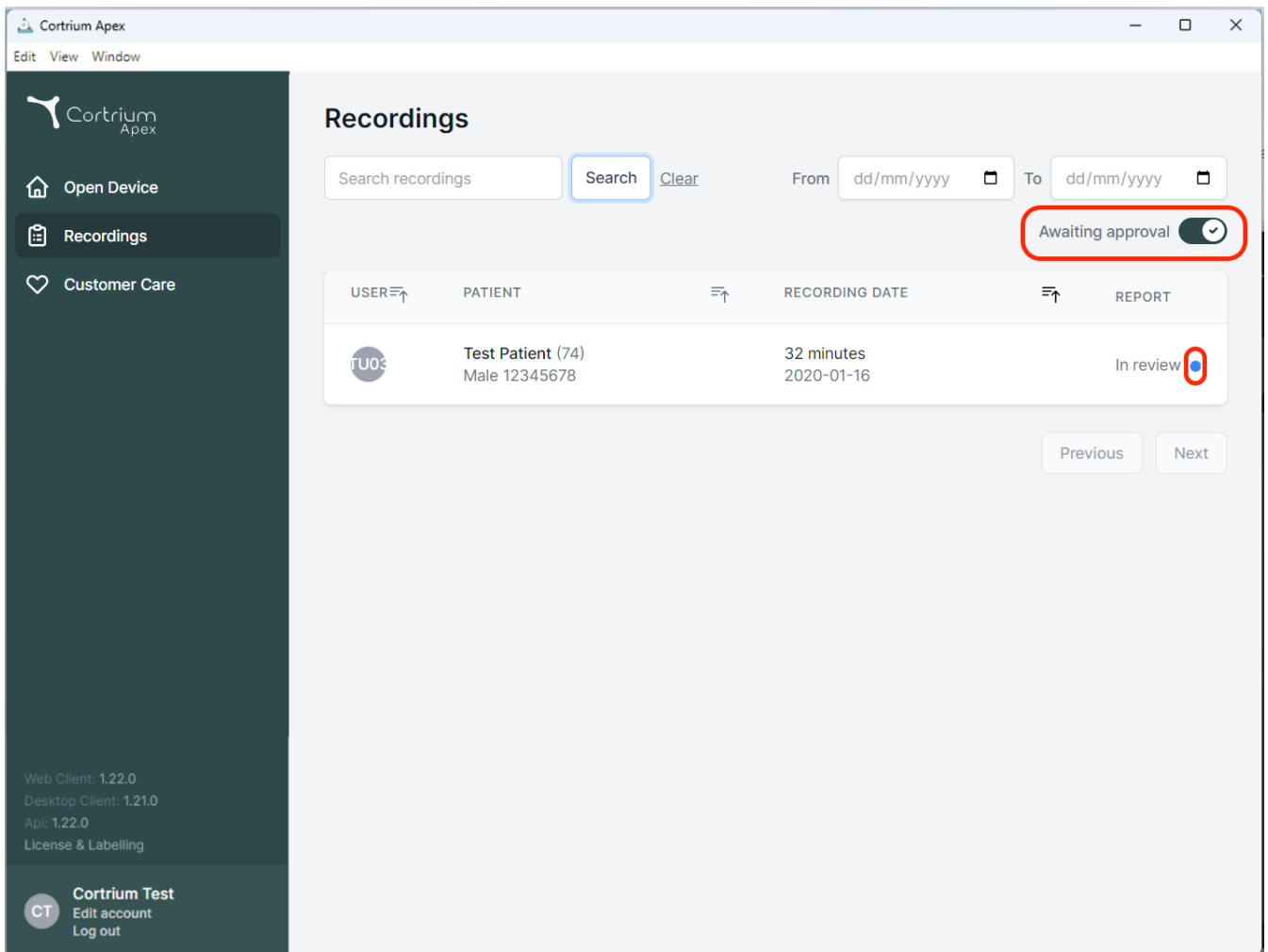
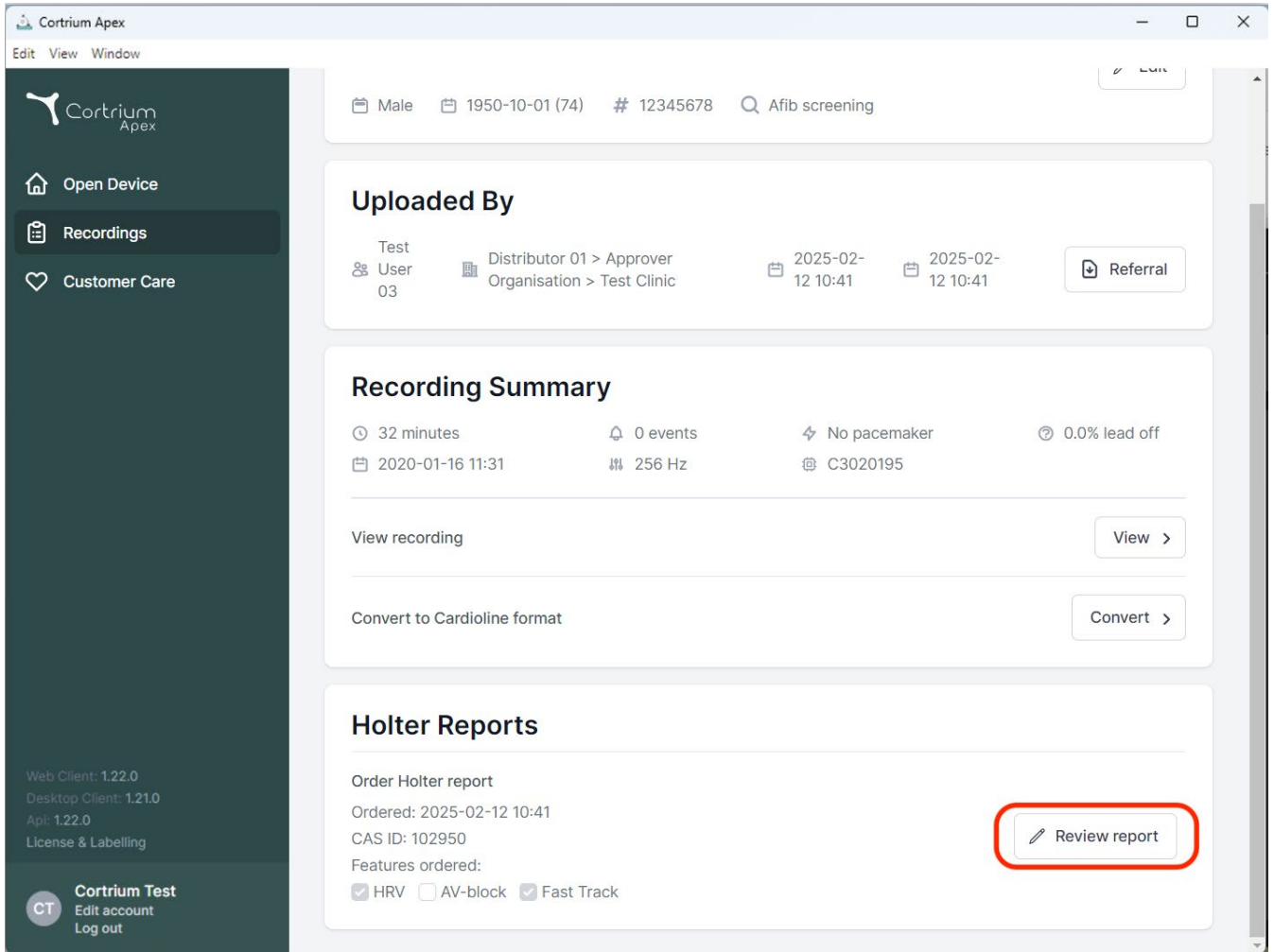
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the "Uploaded By" section, showing Test User 03, Distributor 01 > Approver Organisation > Test Clinic, and upload dates of 2025-02-12 10:41. A "Referral" button is present. The "Recording Summary" section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for "View recording" and "Convert to Cardioline format". The "Holter Reports" section shows an "Order Holter report" with an order date of 2025-02-12 10:41, CAS ID: 102950, and features ordered: HRV, AV-block, and Fast Track. A "Review report" button is highlighted with a red rectangle.

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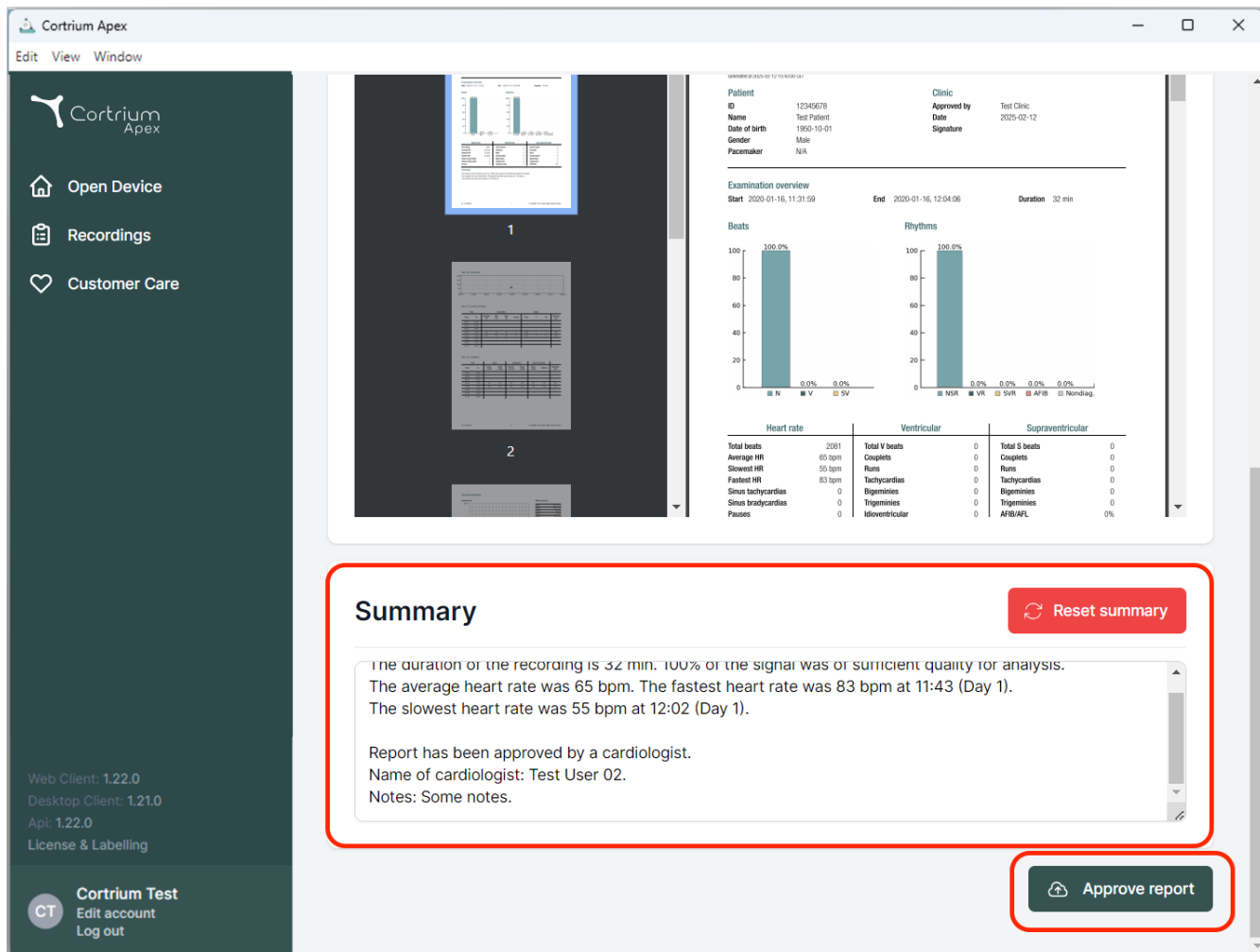The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: "Open Device", "Recordings", and "Customer Care". The main content area shows a patient report for "Test Patient" (ID: 12345678, Date of birth: 1950-10-01, Gender: Male, Pacemaker: N/A). The report includes an "Examination overview" with start/end times and duration, and two bar charts for "Beats" and "Rhythms" showing 100% completion. Below these are tables for "Heart rate", "Ventricular", and "Supraventricular" data. A red box highlights the "Summary" section, which contains the following text:

Summary Reset summary

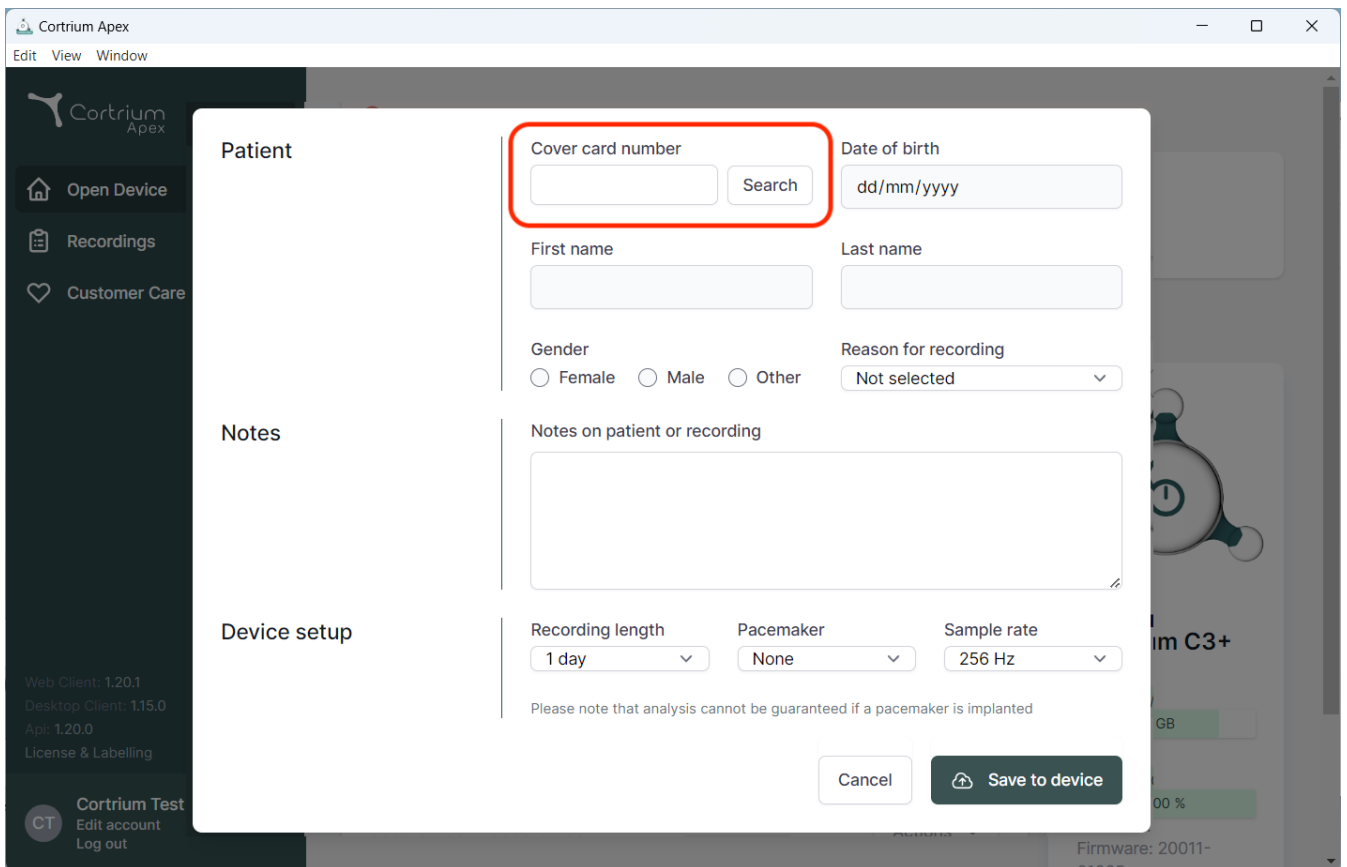
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

At the bottom right, another red box highlights the "Approve report" button.

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The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for patient information. The form includes fields for "Cover card number" (highlighted with a red box), "Date of birth" (format dd/mm/yyyy), "First name", "Last name", "Gender" (radio buttons for Female, Male, Other), and "Reason for recording" (dropdown menu, currently "Not selected"). There is also a large text area for "Notes on patient or recording". Below the form, there are dropdown menus for "Recording length" (set to 1 day), "Pacemaker" (set to None), and "Sample rate" (set to 256 Hz). A note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the form are "Cancel" and "Save to device" buttons. The background shows the main interface with a sidebar containing "Open Device", "Recordings", and "Customer Care" options, and a top menu with "Edit", "View", and "Window".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
E-mail: info@cortrium.com • Hjemmeside: www.cortrium.com

Cortrium Apex UDI(01)05745000379033