Cortrium Apex 1.26

Cortrium Apex 1.26

Kasutusjuhend

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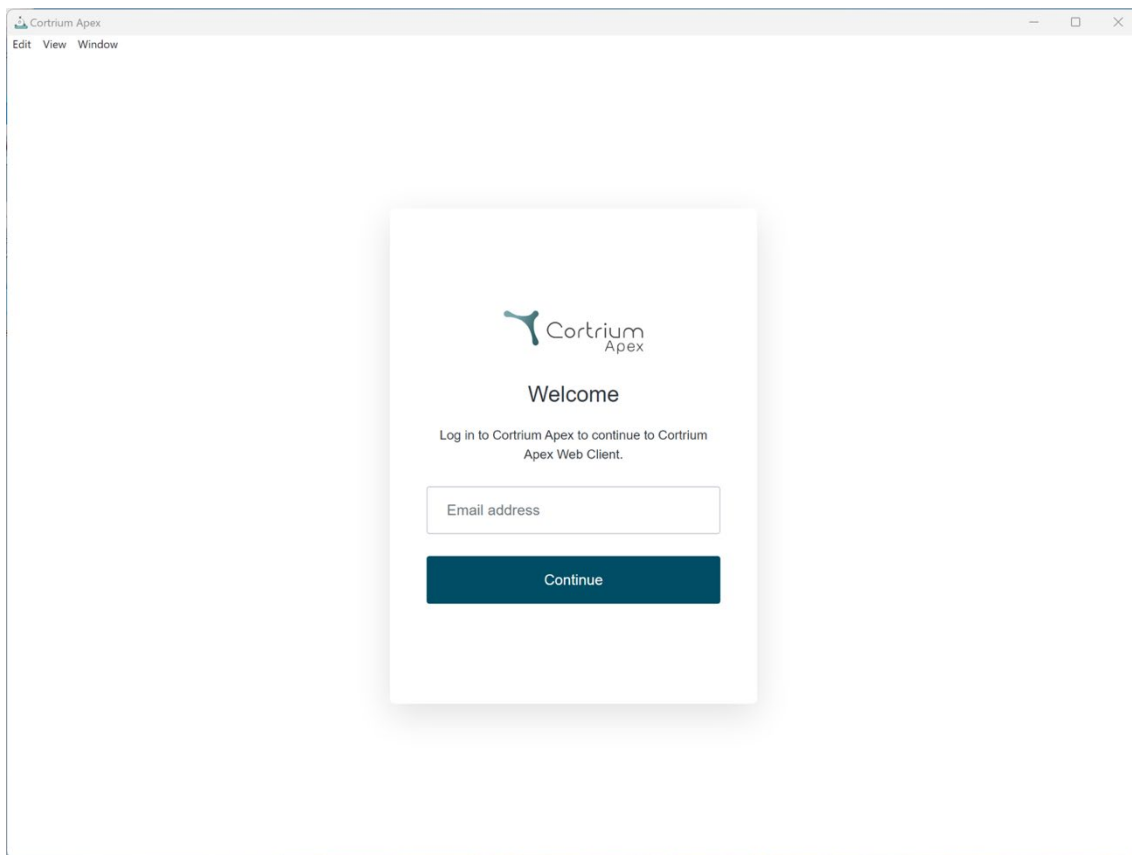
 Cortrium
Apex

Welcome

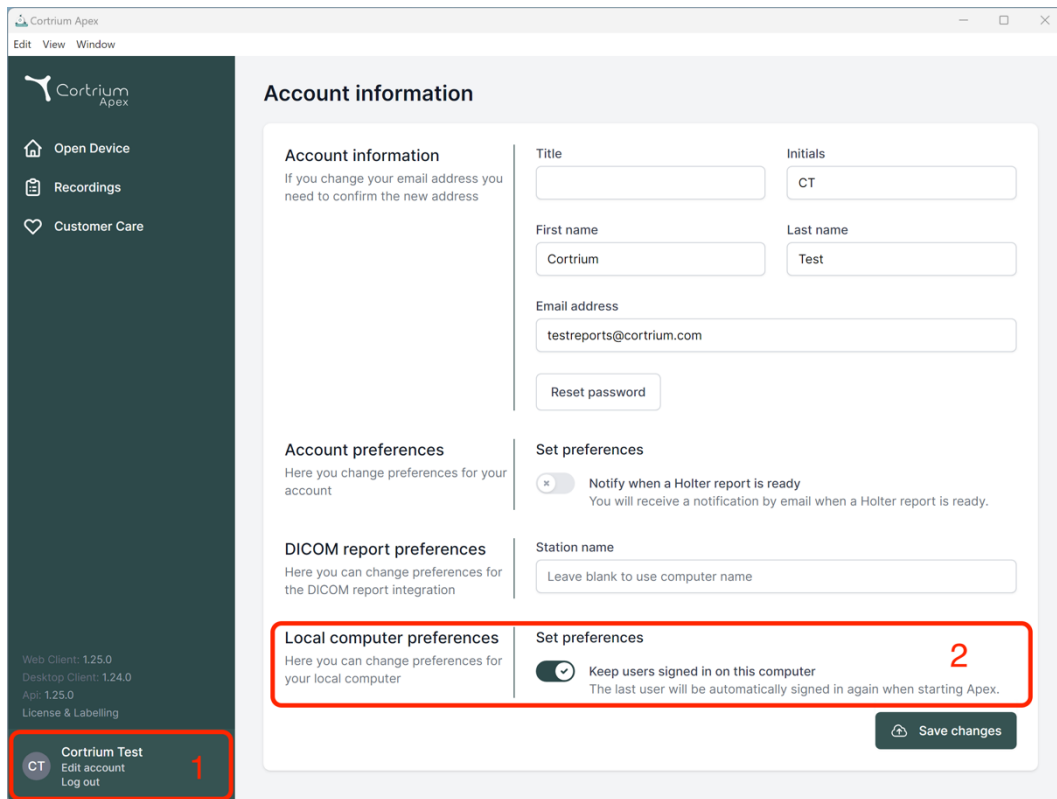
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0, License & Labelling) and a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out'. A red box labeled '1' highlights this user profile section.

The main content area is titled 'Account information' and contains several sections:

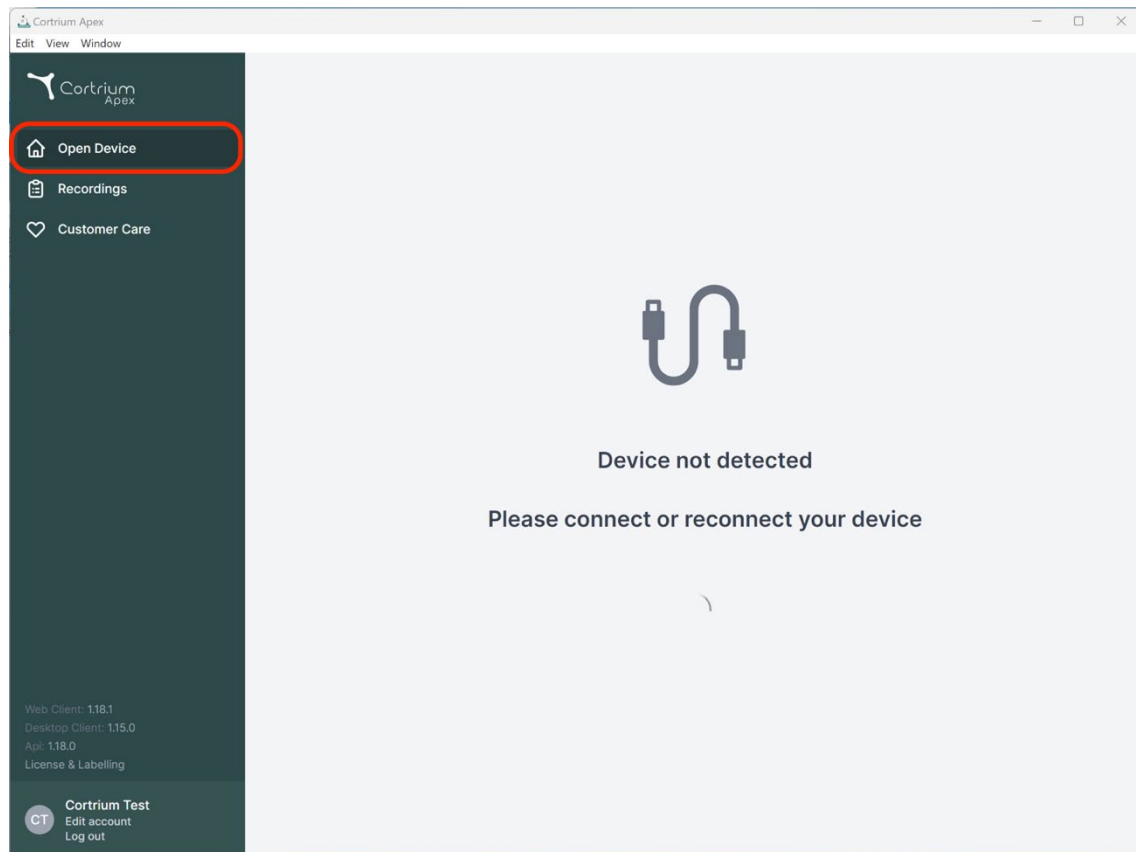
- Account information:** Includes fields for Title, Initials (CT), First name (Cortrium), Last name (Test), and Email address (testreports@cortrium.com). A 'Reset password' button is also present.
- Account preferences:** A section for changing account preferences.
- DICOM report preferences:** A section for changing preferences for DICOM report integration.
- Local computer preferences:** A section for changing preferences for the local computer. A red box labeled '2' highlights this section.
- Set preferences:** Includes a toggle for 'Notify when a Holter report is ready' and a 'Station name' field.
- Local computer preferences (continued):** Includes a toggle for 'Keep users signed in on this computer' with the note 'The last user will be automatically signed in again when starting Apex.' A red box labeled '2' also highlights this toggle.

A 'Save changes' button is located at the bottom right of the main content area.

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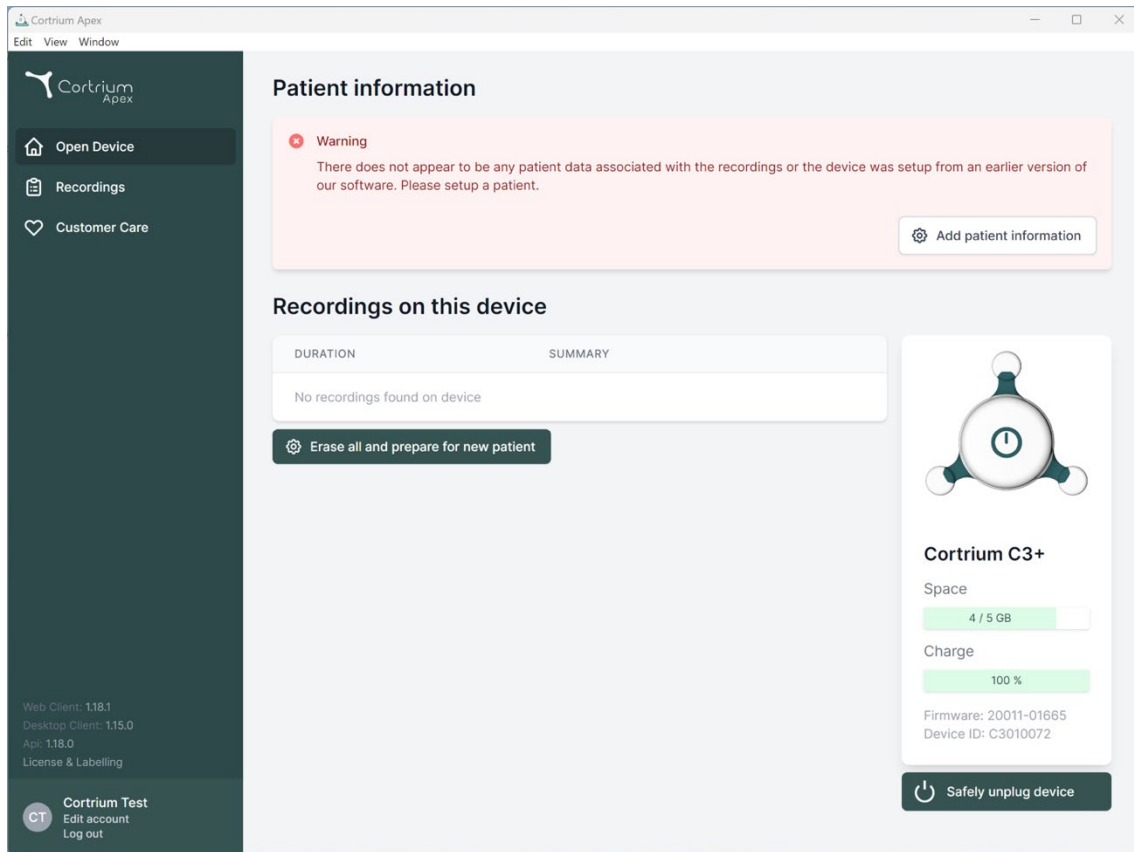
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning
This will erase all recordings on the device

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
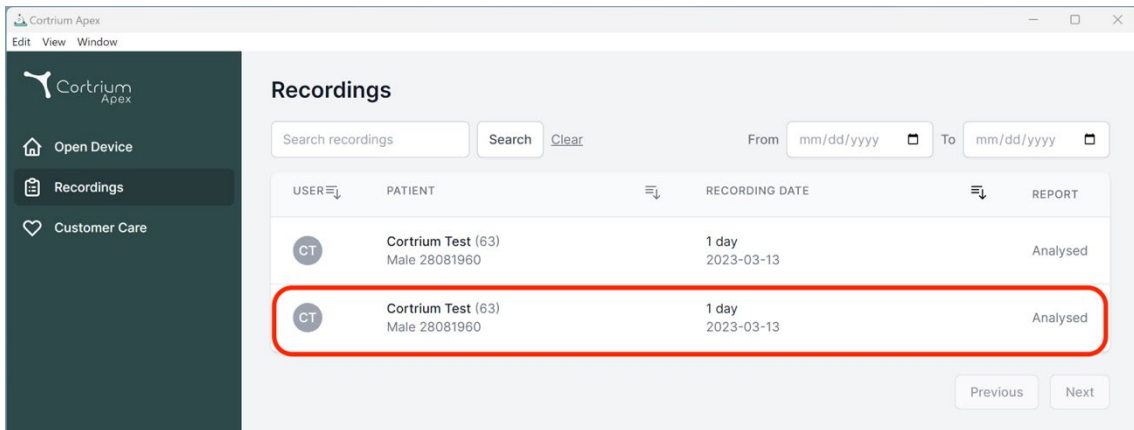
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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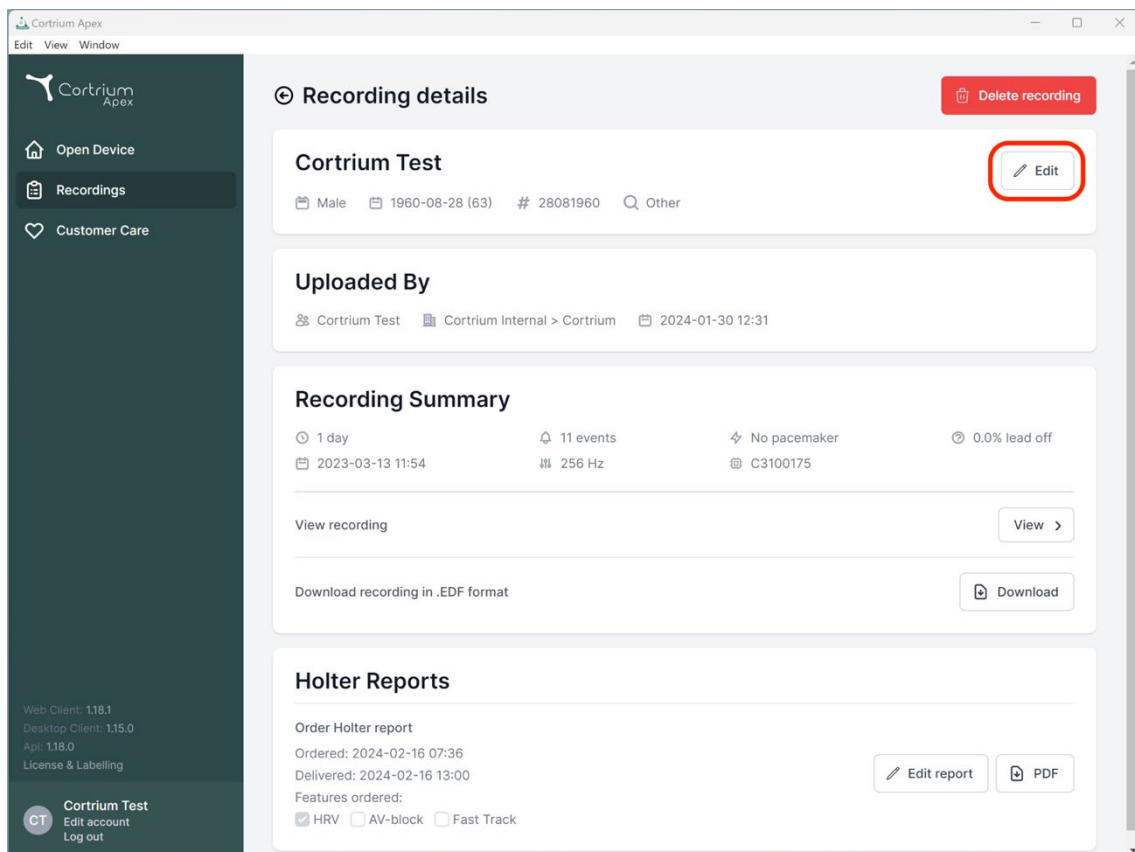Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

[View recording](#) [View >](#)

[Download recording in .EDF format](#) [Download](#)

Holter Reports

Order Holter report
Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00
Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track [Edit report](#) [PDF](#)

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
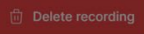
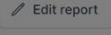
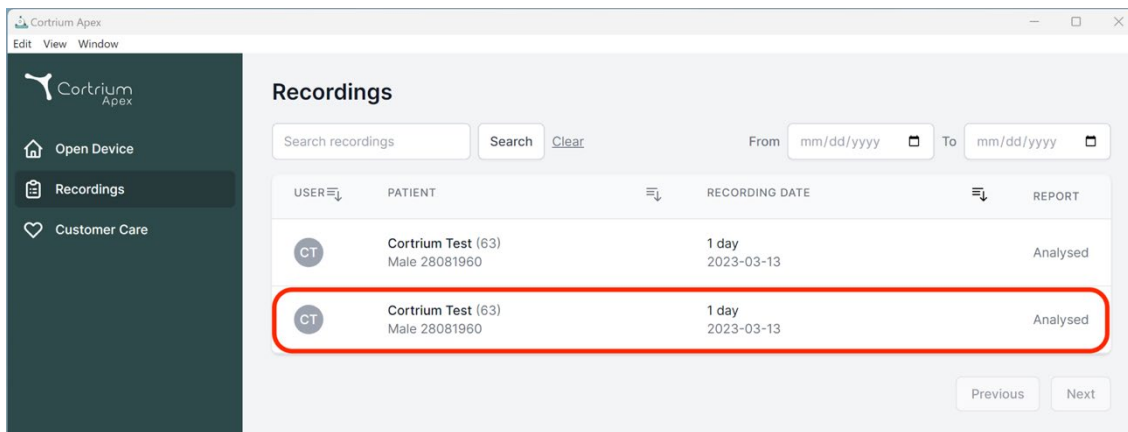
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

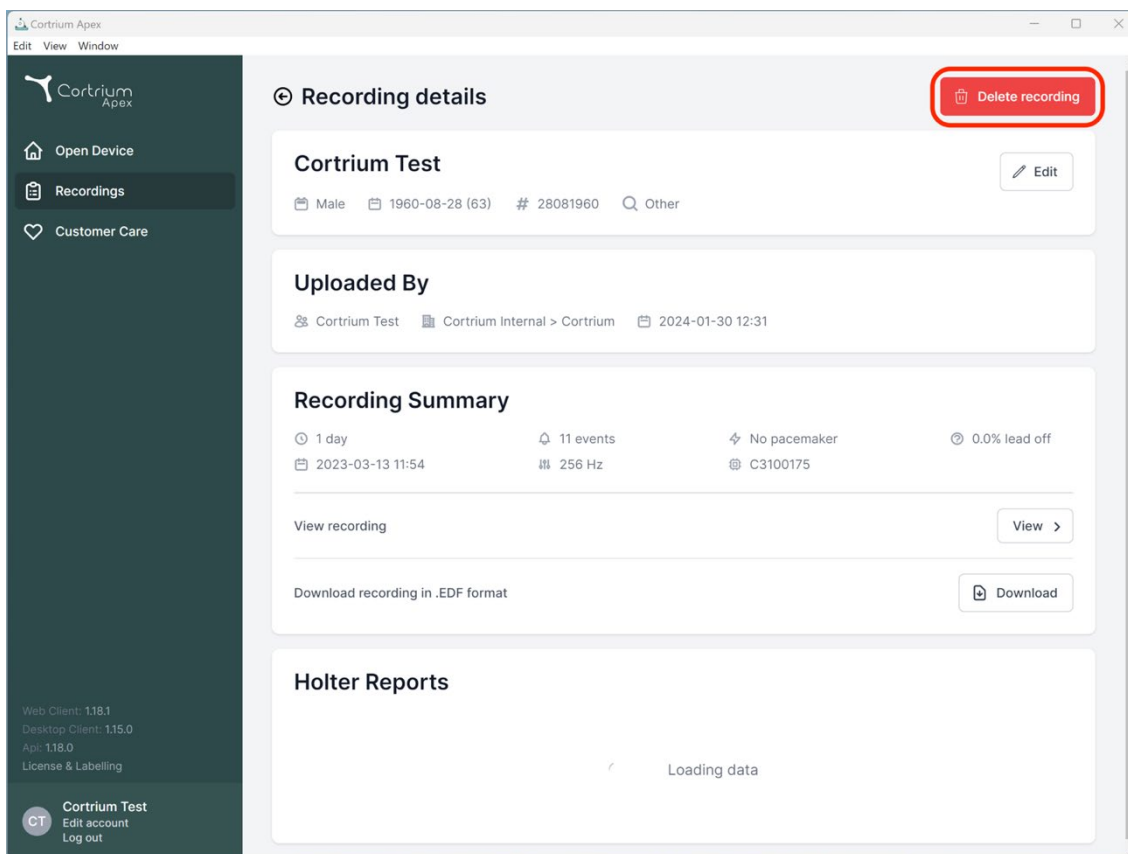Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Loading data

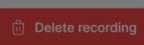
Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Recording details 

Cortrium Test 

 Male  1960-08-28 (63)  # 28081960  Other

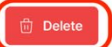
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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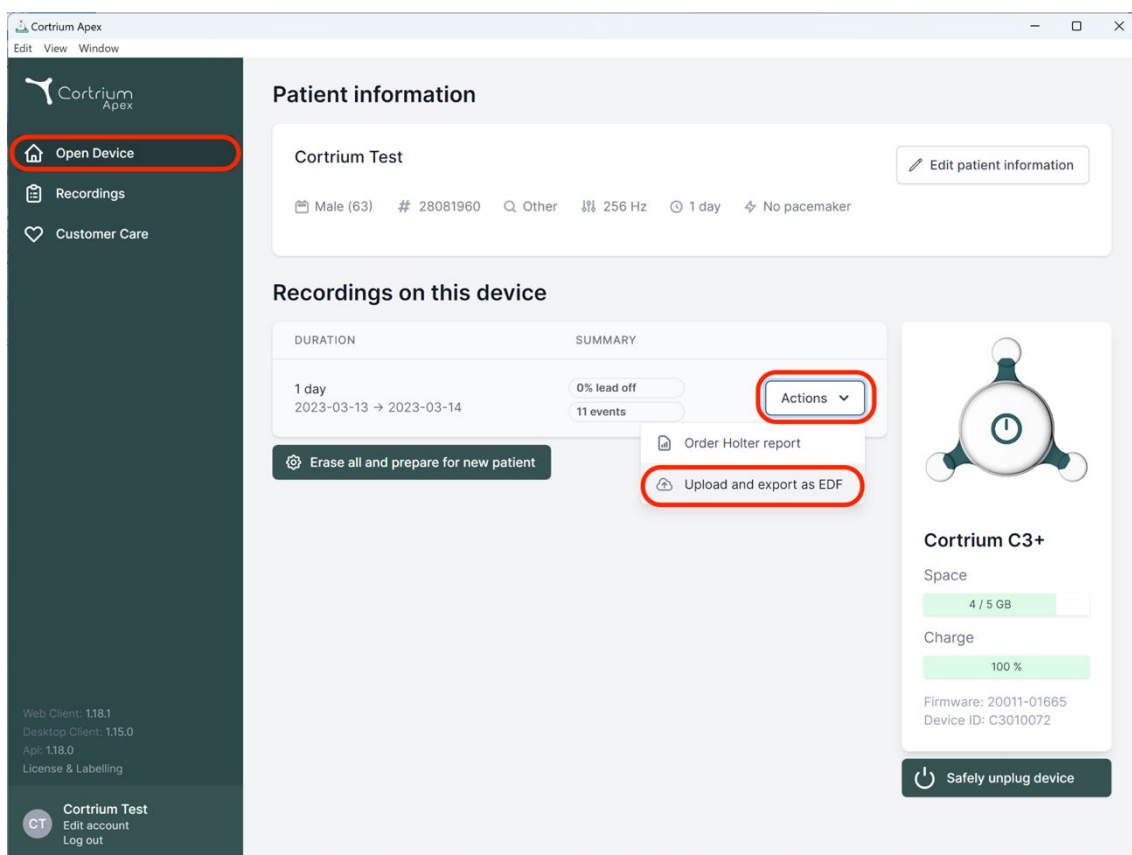
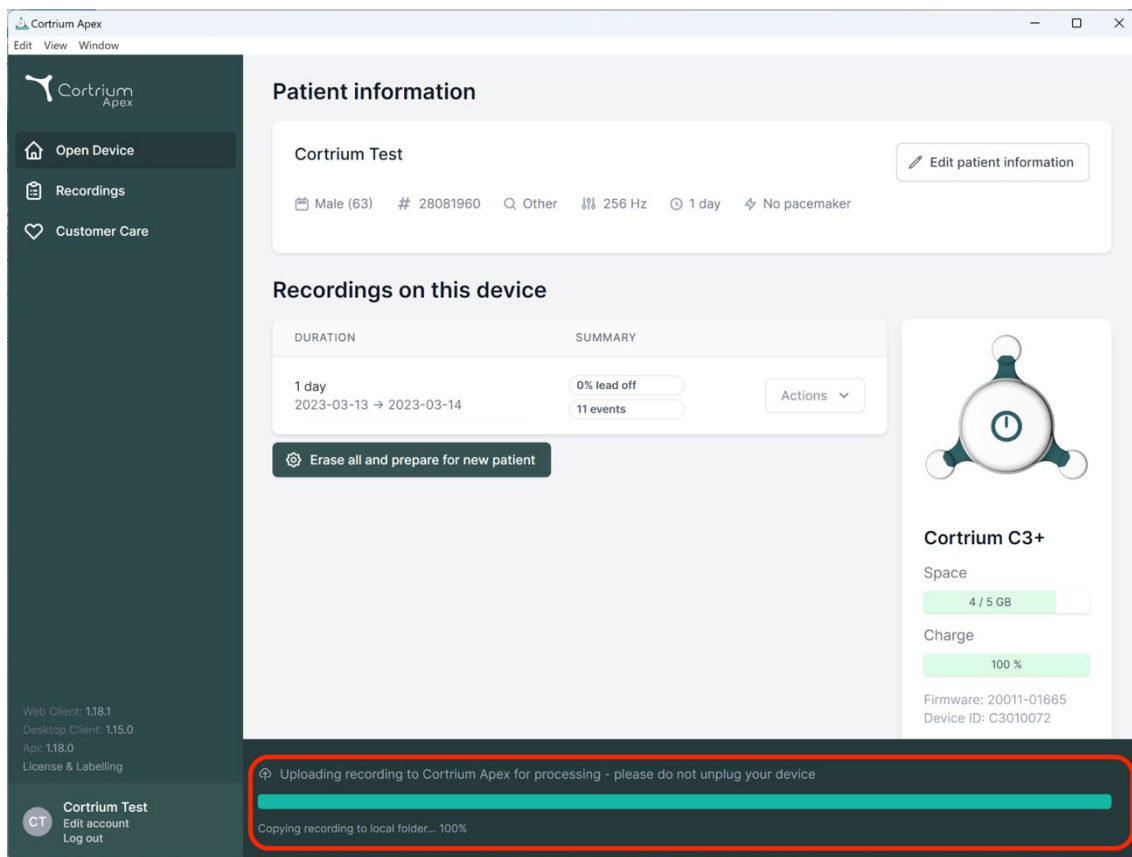
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

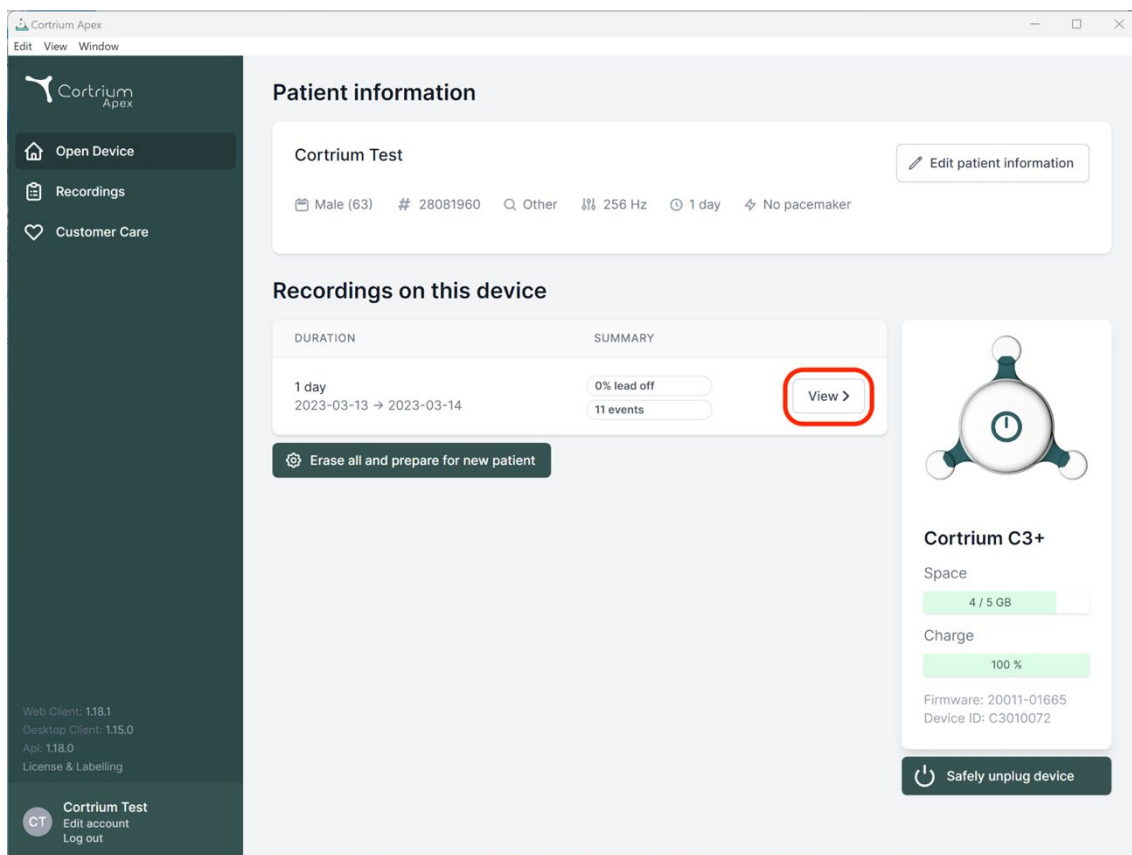
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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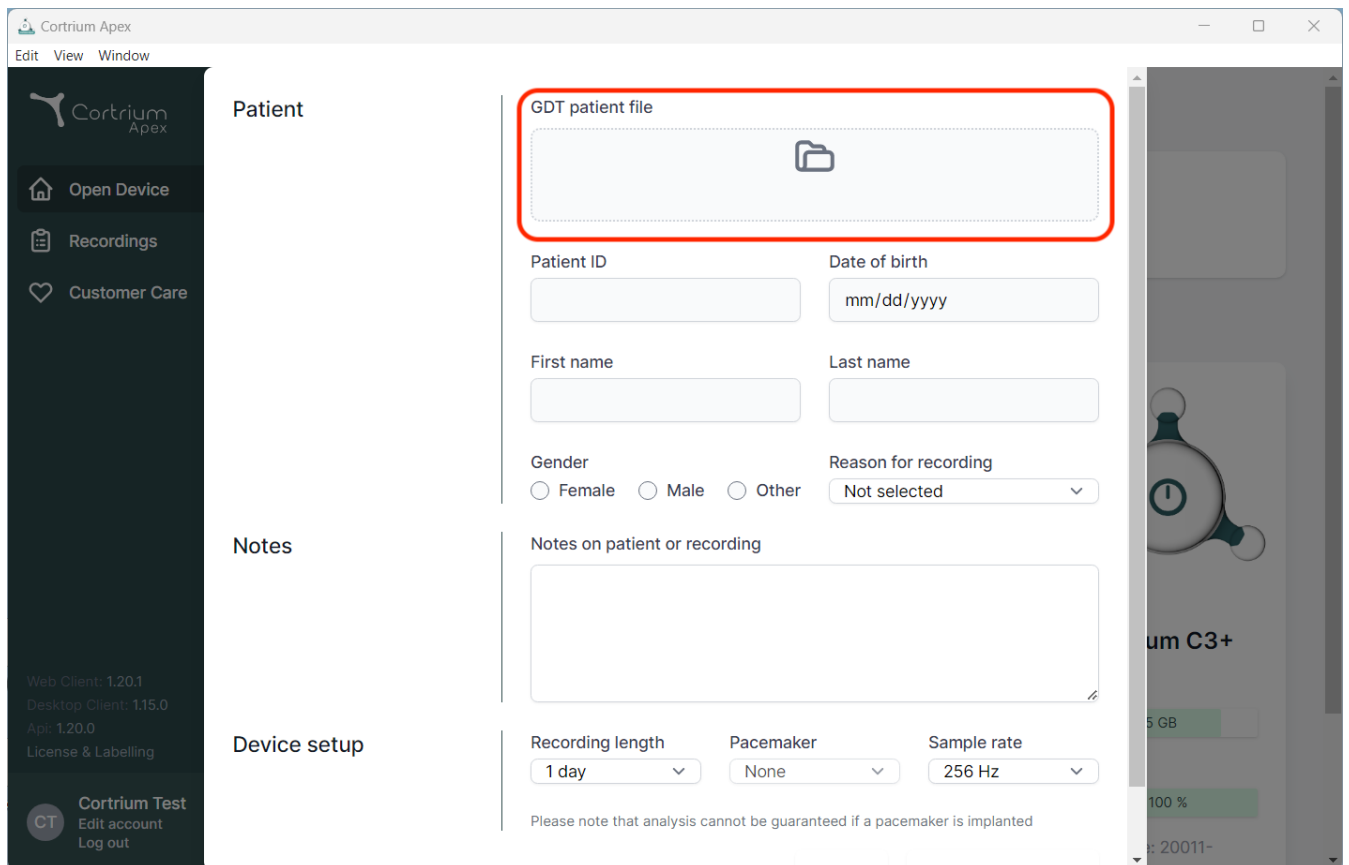
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a user profile section for "Cortrium Test" with options to "Edit account" and "Log out".

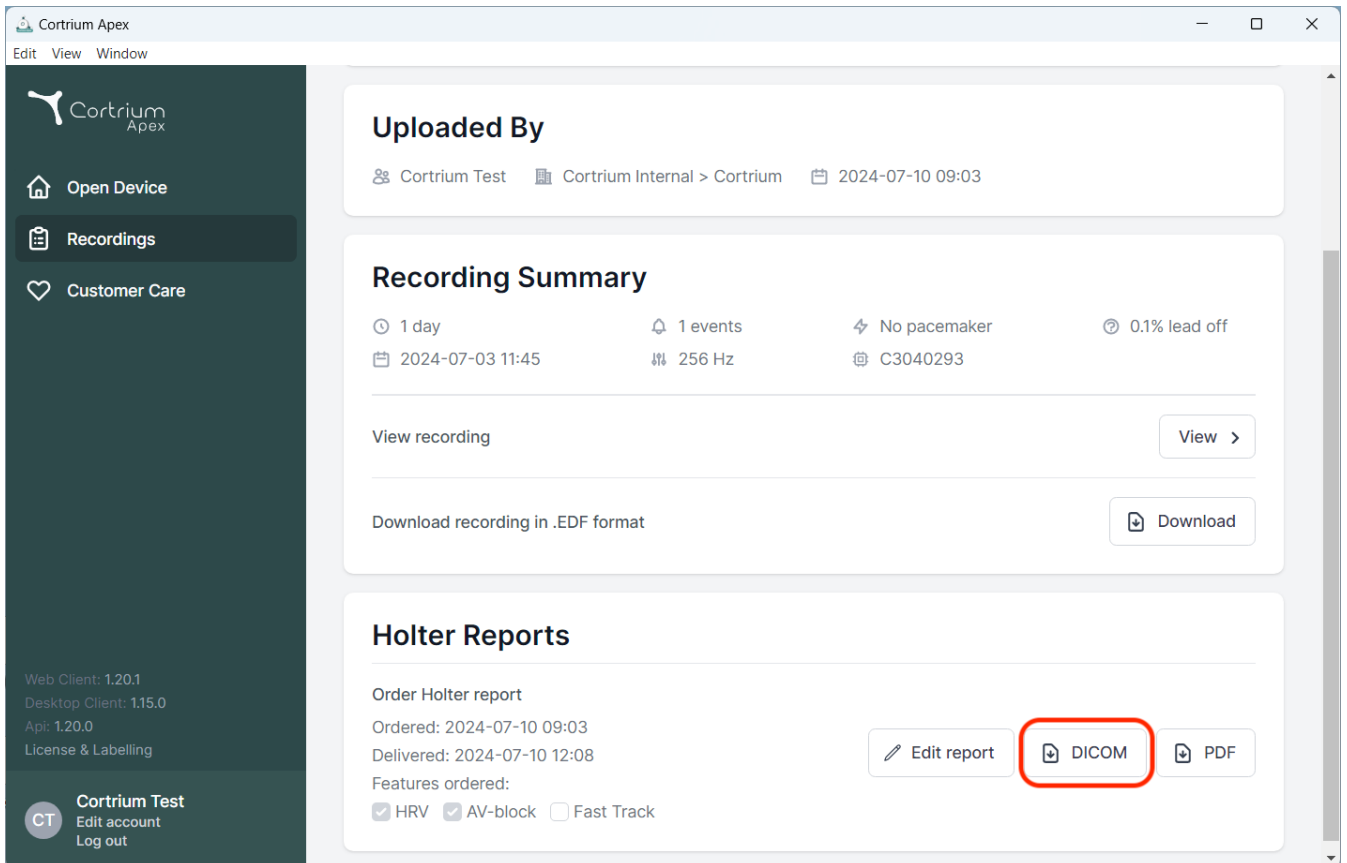
The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

- Patient section:** Contains a "GDT patient file" upload area (highlighted with a red box) with a folder icon. Below this are input fields for "Patient ID", "Date of birth" (format mm/dd/yyyy), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown for "Reason for recording" (currently "Not selected").
- Notes section:** Includes a text area labeled "Notes on patient or recording".
- Device setup section:** Features dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device model, "um C3+", is visible, showing a battery level of "100 %" and a date "20011-".

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

Uploaded By
Cortrium Test | Cortrium Internal > Cortrium | 2024-07-10 09:03

Recording Summary
1 day | 1 events | No pacemaker | 0.1% lead off
2024-07-03 11:45 | 256 Hz | C3040293

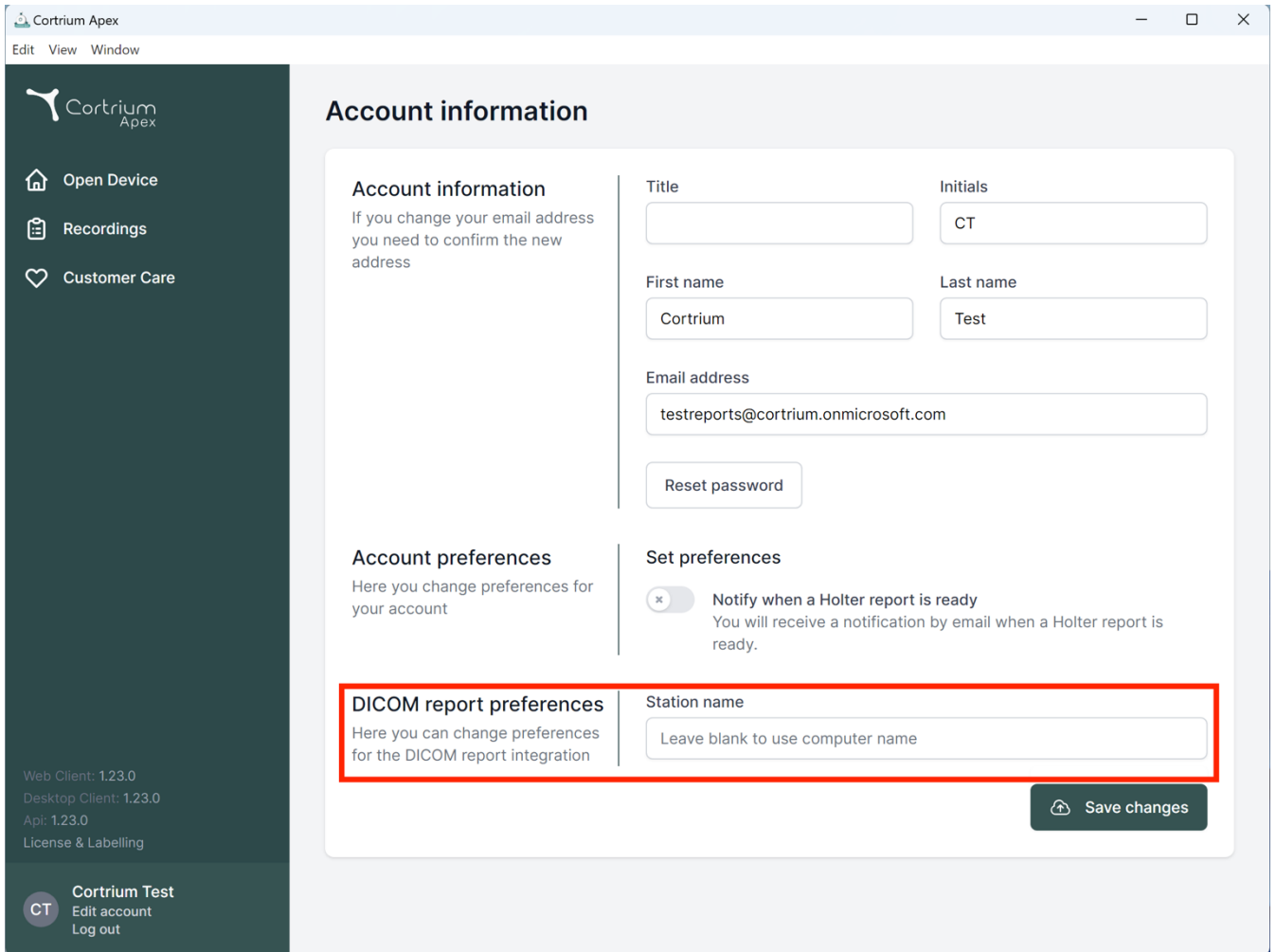
View recording [View >]
Download recording in .EDF format [Download]

Holter Reports
Order Holter report
Ordered: 2024-07-10 09:03
Delivered: 2024-07-10 12:08
Features ordered:
☒ HRV ☒ AV-block ☐ Fast Track

Buttons: [Edit report] [DICOM] [PDF]. The 'DICOM' button is highlighted with a red rectangle.

Footer information:
Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling
CT Cortrium Test
Edit account
Log out

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Cortrium Apex
Edit View Window

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

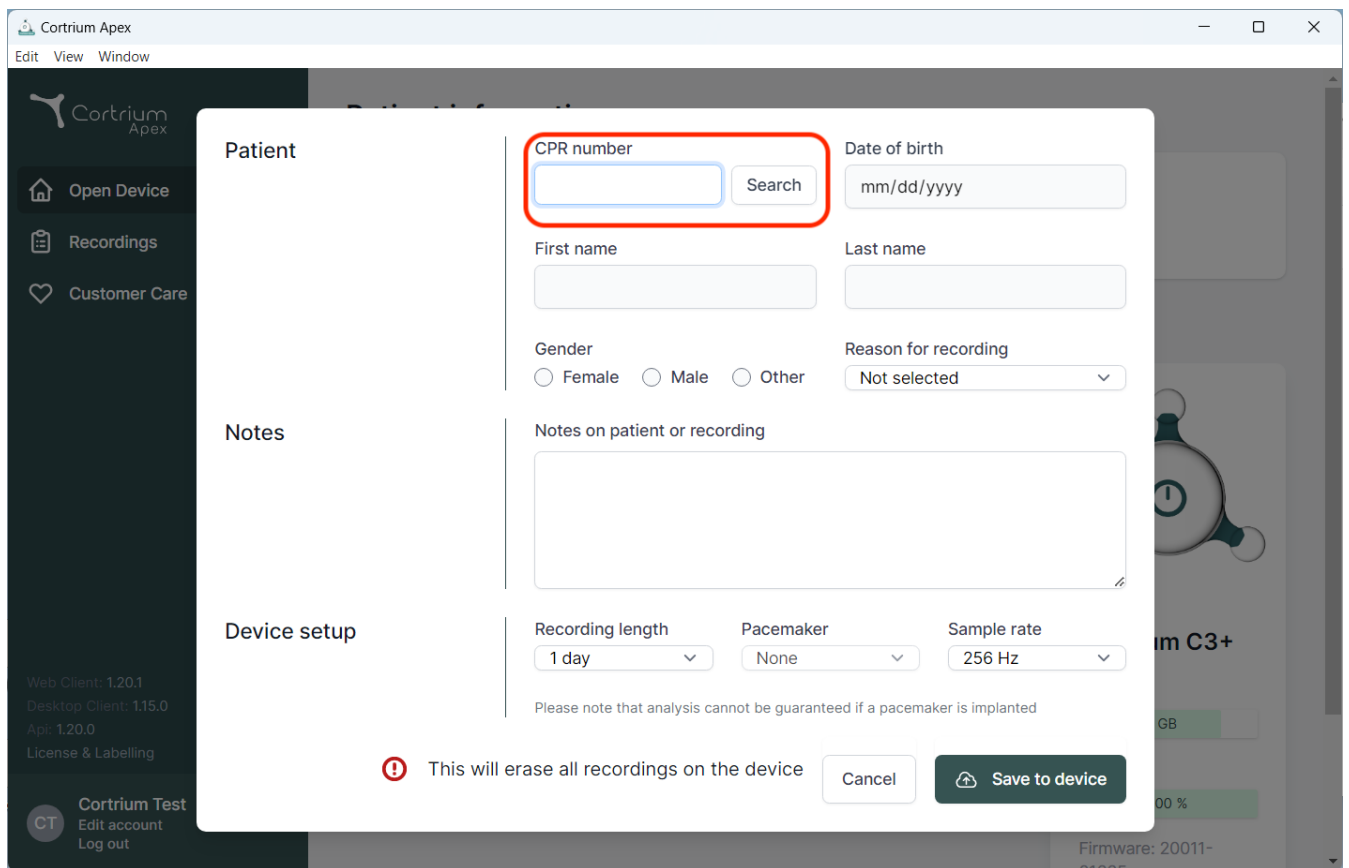
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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Pacemaker

Sample rate

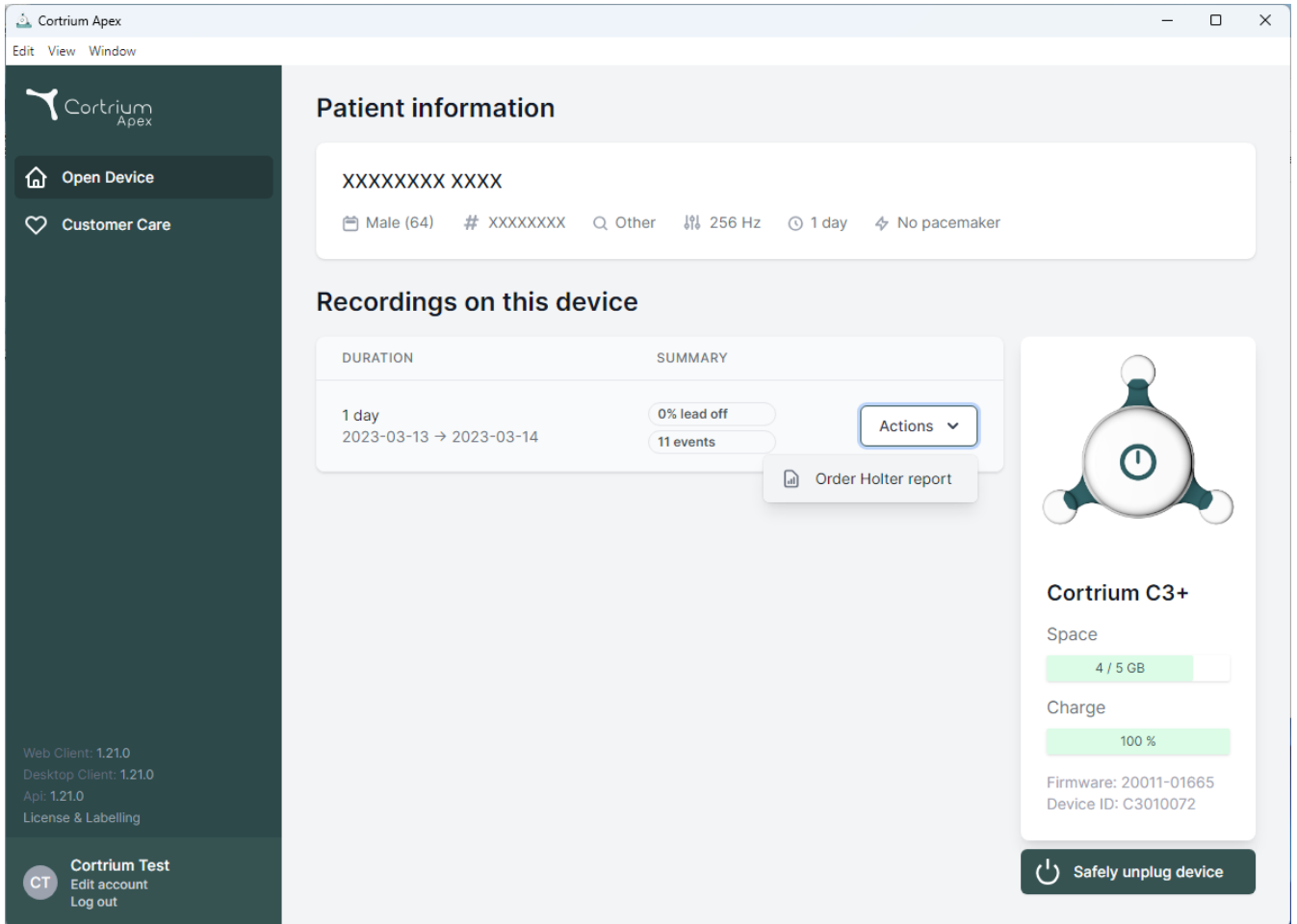
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' options.

The main content area is titled 'Patient information' and displays a patient ID 'XXXXXXXX XXXX'. Below this, patient details are shown: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for a '1 day' recording from '2023-03-13 → 2023-03-14'. The summary for this recording shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side of the interface, there is a section for the 'Cortrium C3+' device. It includes a diagram of the device, its name, and status indicators for 'Space' (4 / 5 GB) and 'Charge' (100 %). Below these, the 'Firmware: 20011-01665' and 'Device ID: C3010072' are listed. At the bottom right, there is a 'Safely unplug device' button with a power icon.

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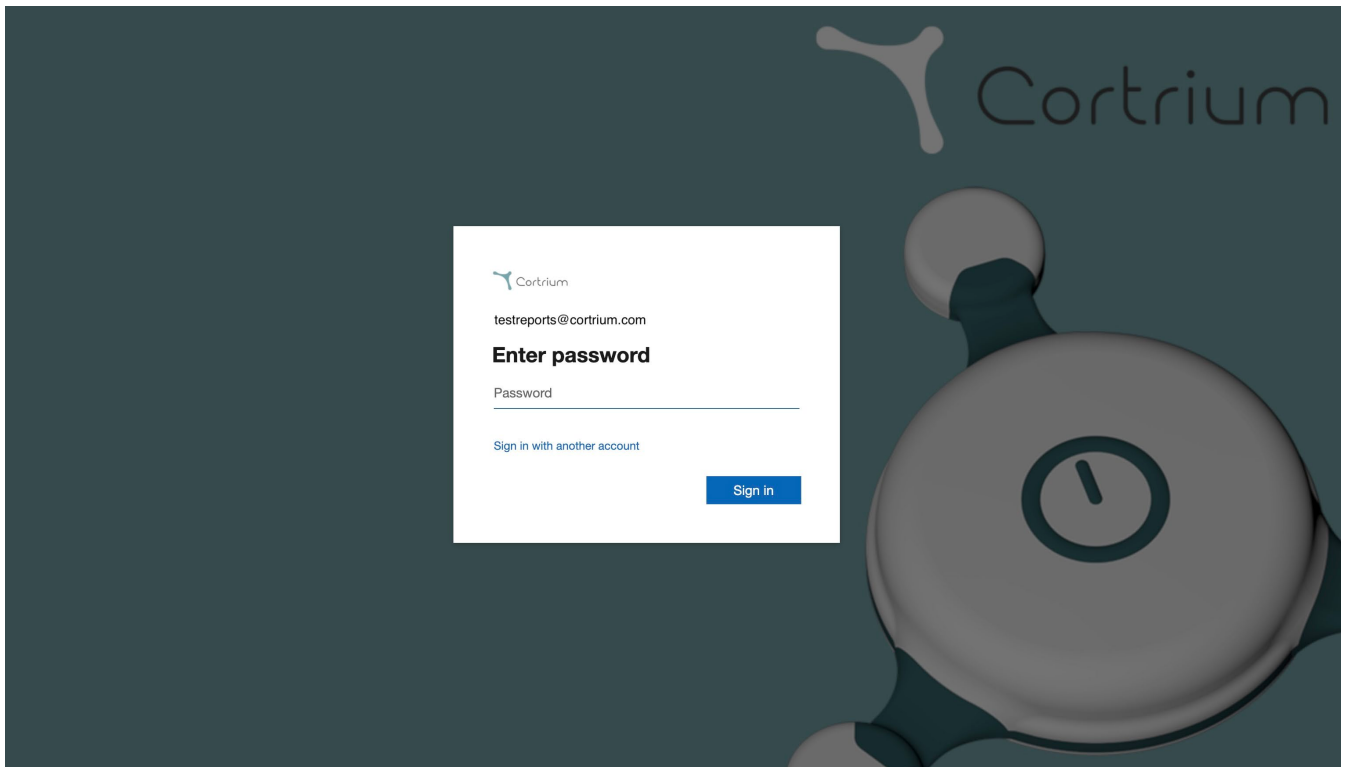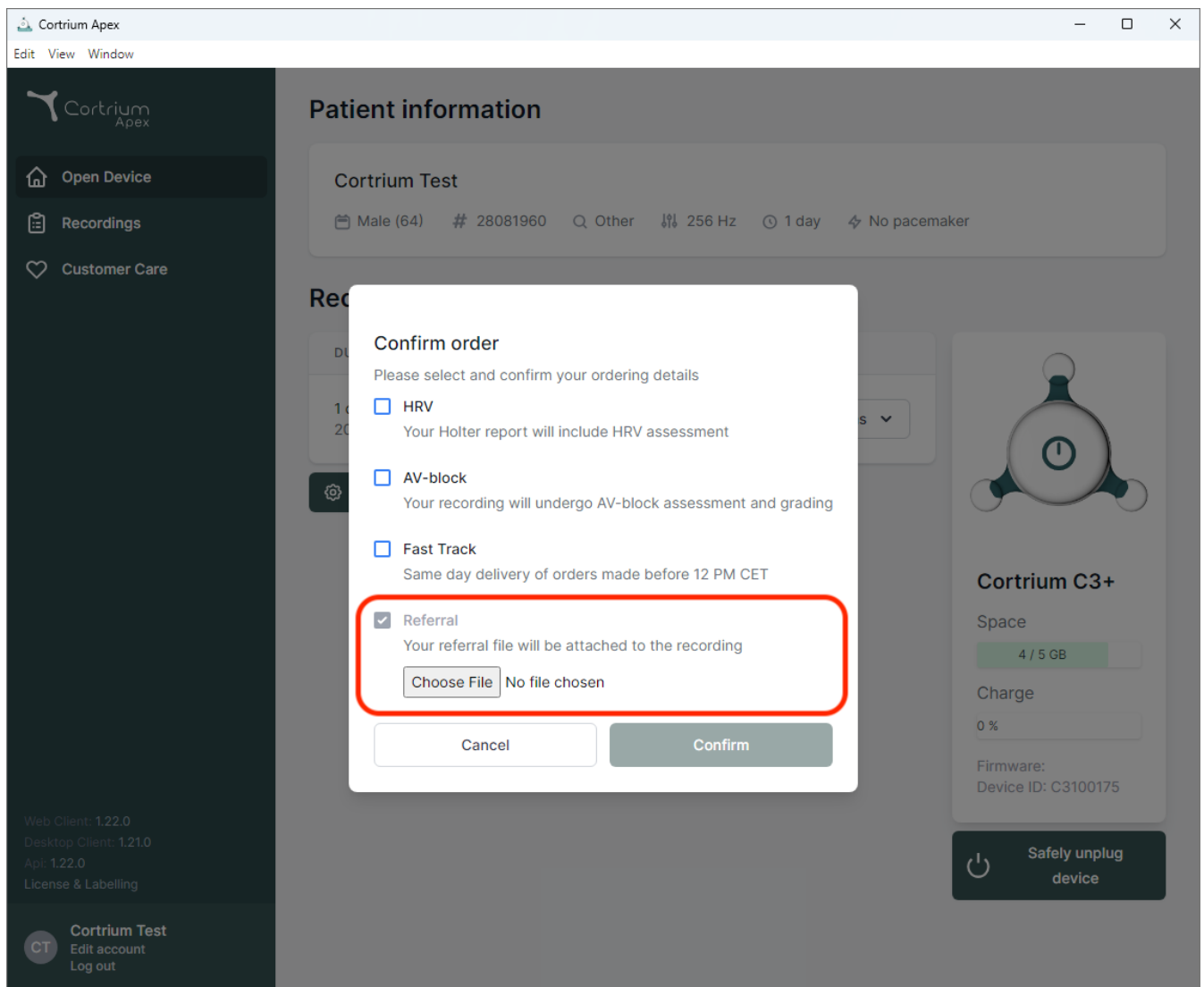
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
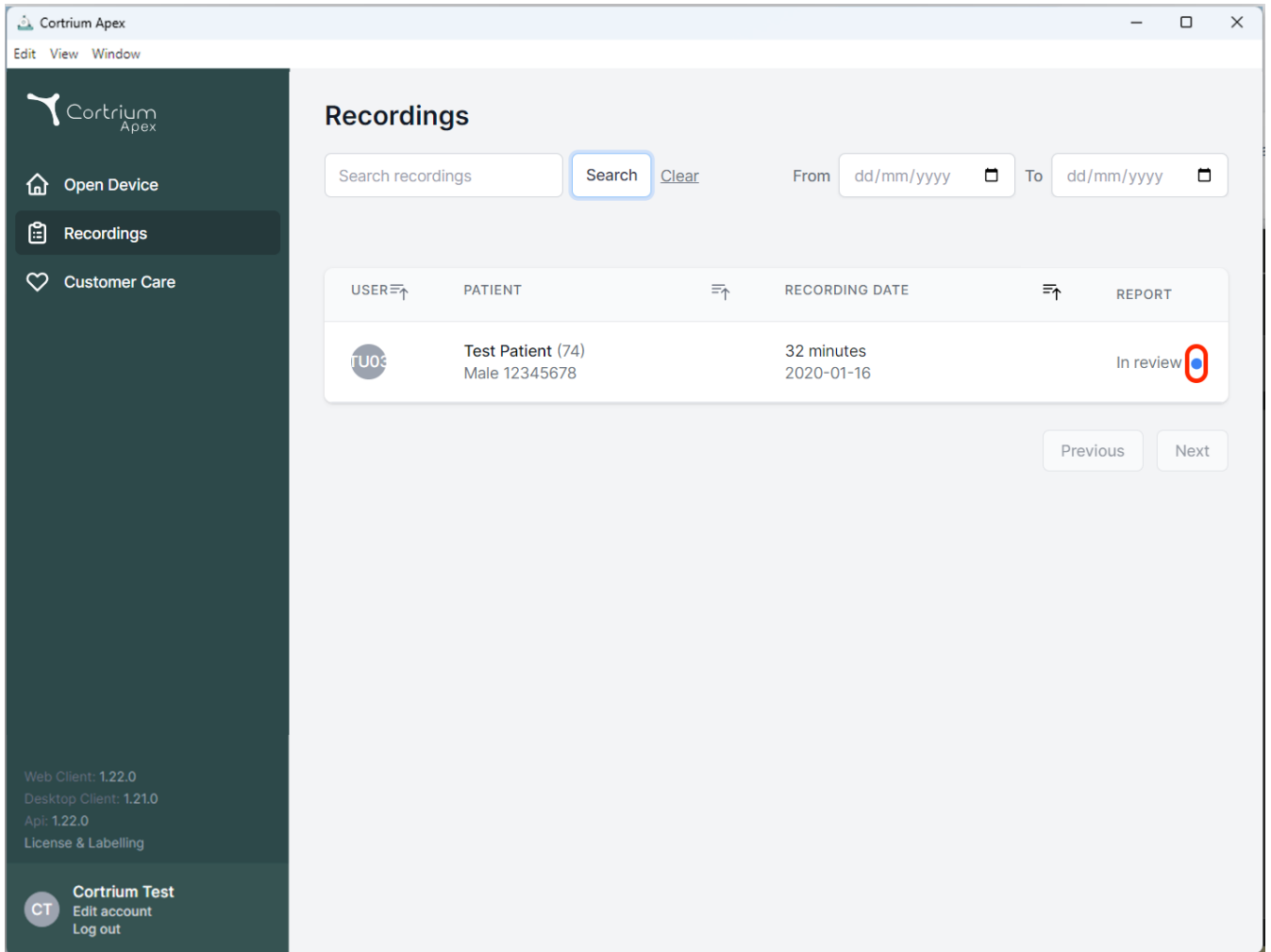
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: "Open Device", "Recordings" (selected), and "Customer Care". The main area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for user "TU03", patient "Test Patient (74) Male 12345678", with a duration of "32 minutes" and a date of "2020-01-16". The status "In review" is shown next to a blue dot icon, which is circled in red. At the bottom right of the table are "Previous" and "Next" buttons. The footer of the sidebar shows version information: "Web Client: 1.22.0", "Desktop Client: 1.21.0", "Api: 1.22.0", and "License & Labelling". The user "Cortrium Test" is logged in, with options to "Edit account" and "Log out".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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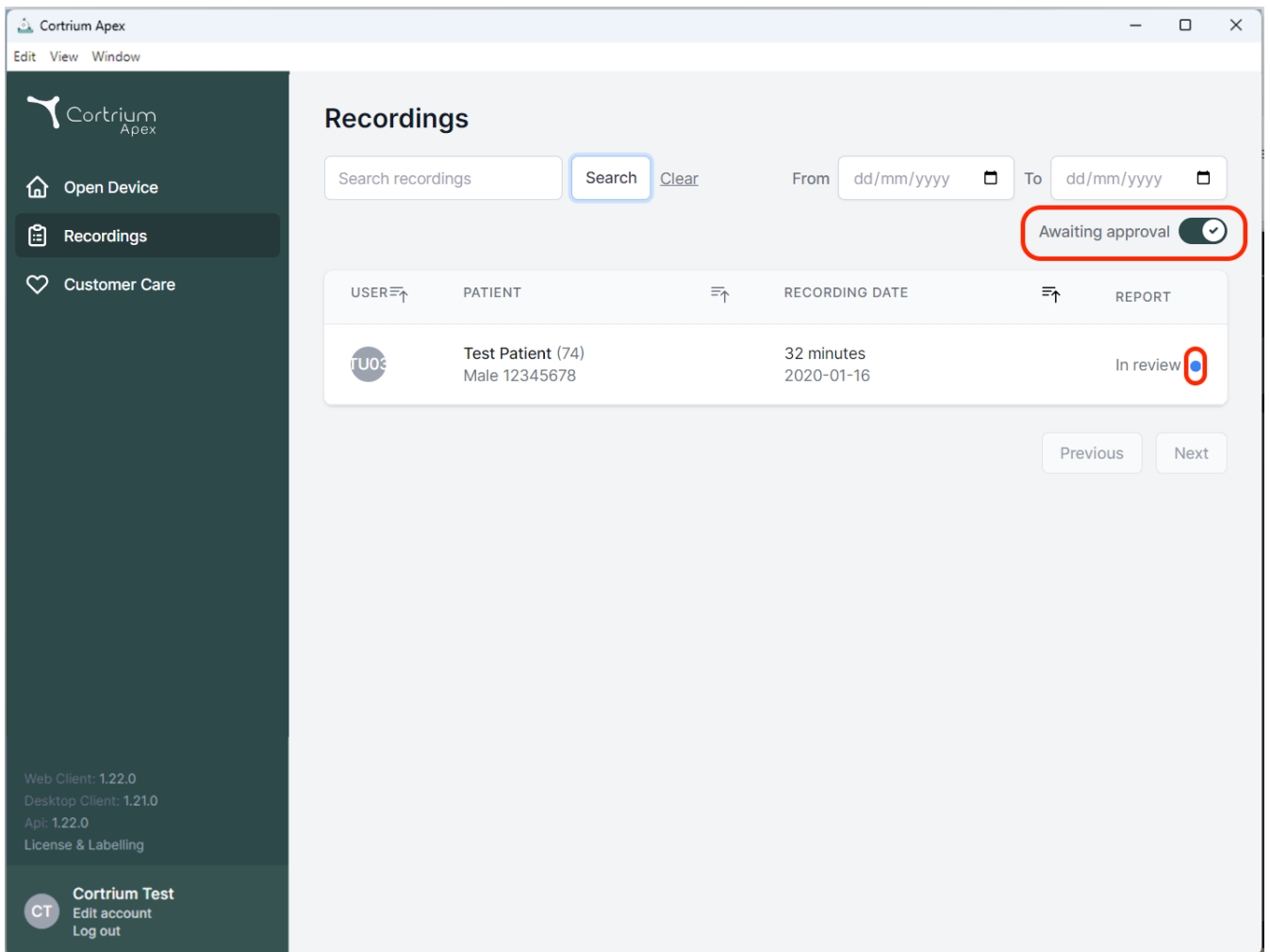
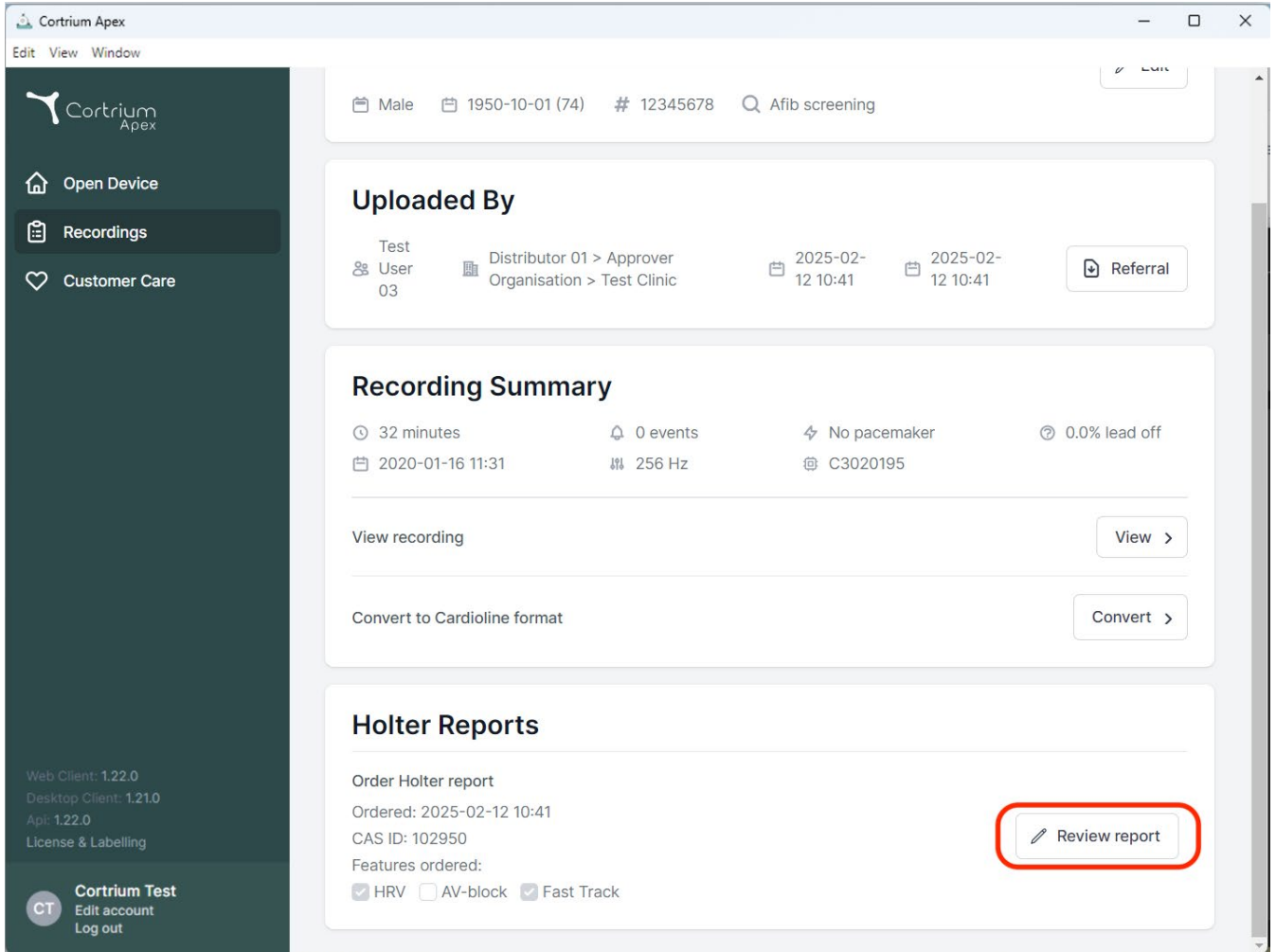
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41. A 'Referral' button is present. The 'Recording Summary' section displays: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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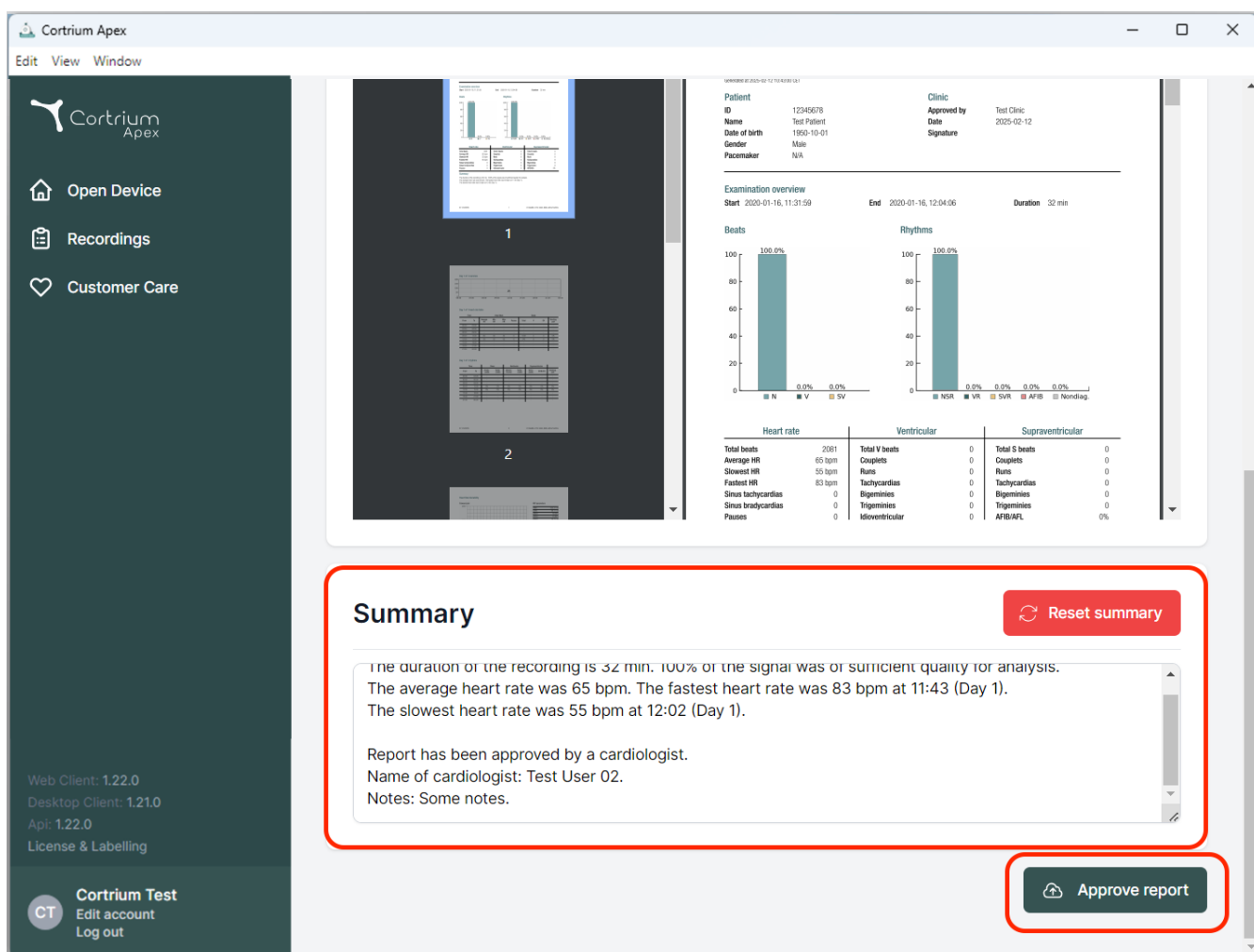Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
App: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

Beats
100.0%
0.0%
0.0%

Rhythms
100.0%
0.0%
0.0%
0.0%
0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Midventricular	0	AF/IB/AF/FL	0%

Summary [Reset summary](#)

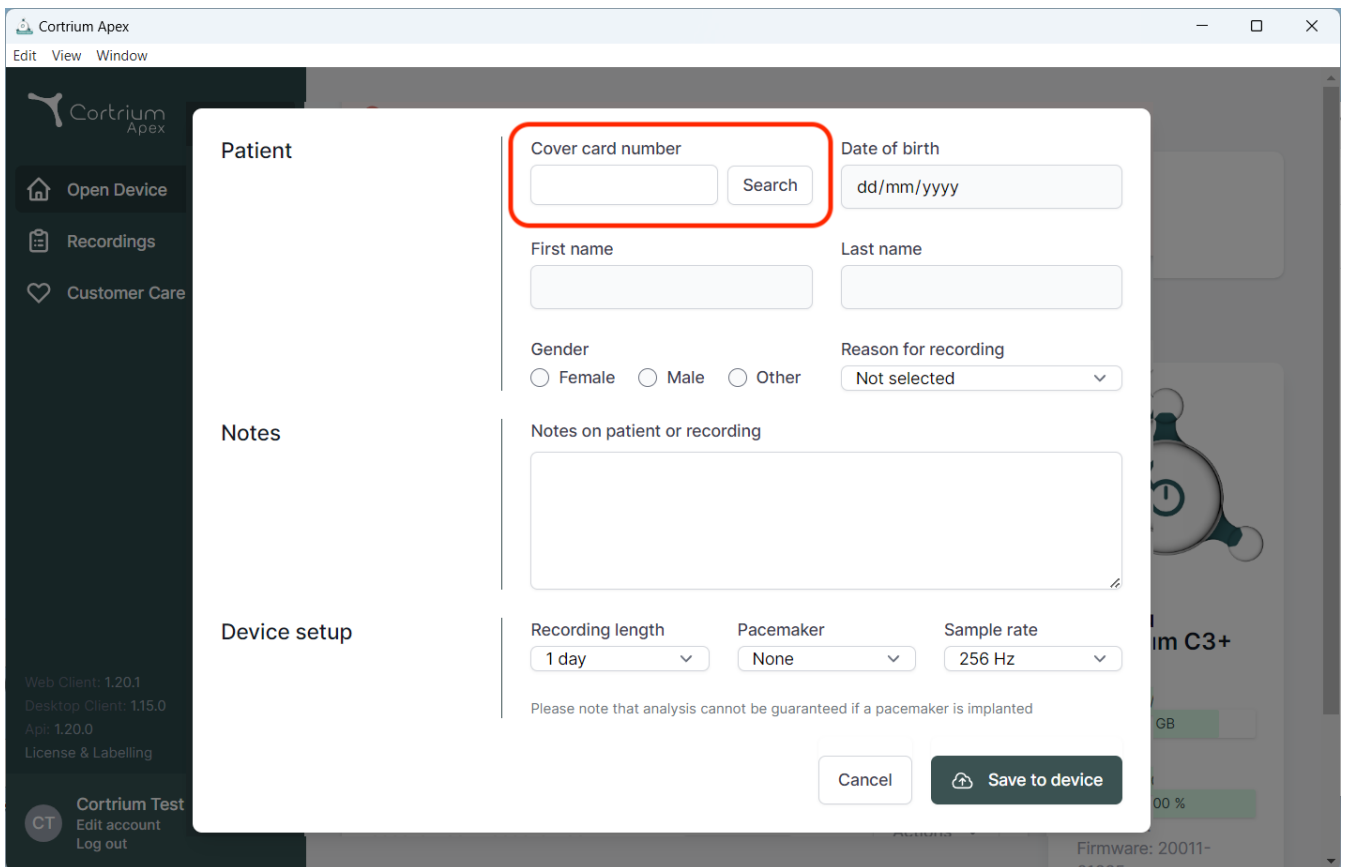
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, allowing for patient search and data entry. The form includes the following fields and options:

- Cover card number:** A text input field with a "Search" button next to it. This section is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Includes dropdowns for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup fields, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the modal, there are "Cancel" and "Save to device" buttons.

The background interface shows a sidebar with "Open Device", "Recordings", and "Customer Care" options, and a main area displaying device information for "im C3+", including storage (GB) and battery (00 %) levels.

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Taani
E-post: info@cortrium.com • Veebisait: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47