Cortrium Apex 1.26

Cortrium Apex 1.26

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
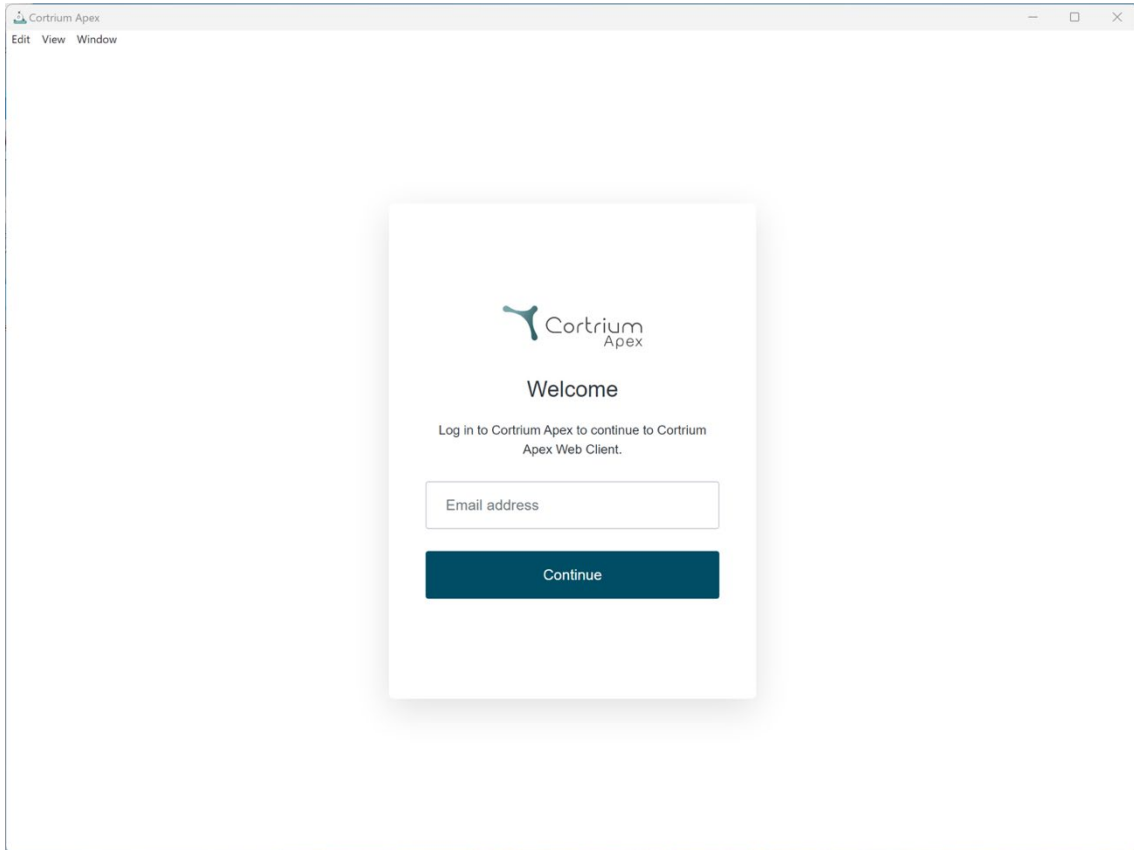
The screenshot shows a web browser window titled "Cortrium Apex". The browser's address bar is empty, and the menu bar shows "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle drop shadow. At the top of the form is the Cortrium Apex logo. Below the logo is the heading "Welcome". Underneath the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." Below this text is a text input field with the placeholder "Email address". At the bottom of the form is a dark blue button with the text "Continue" in white.

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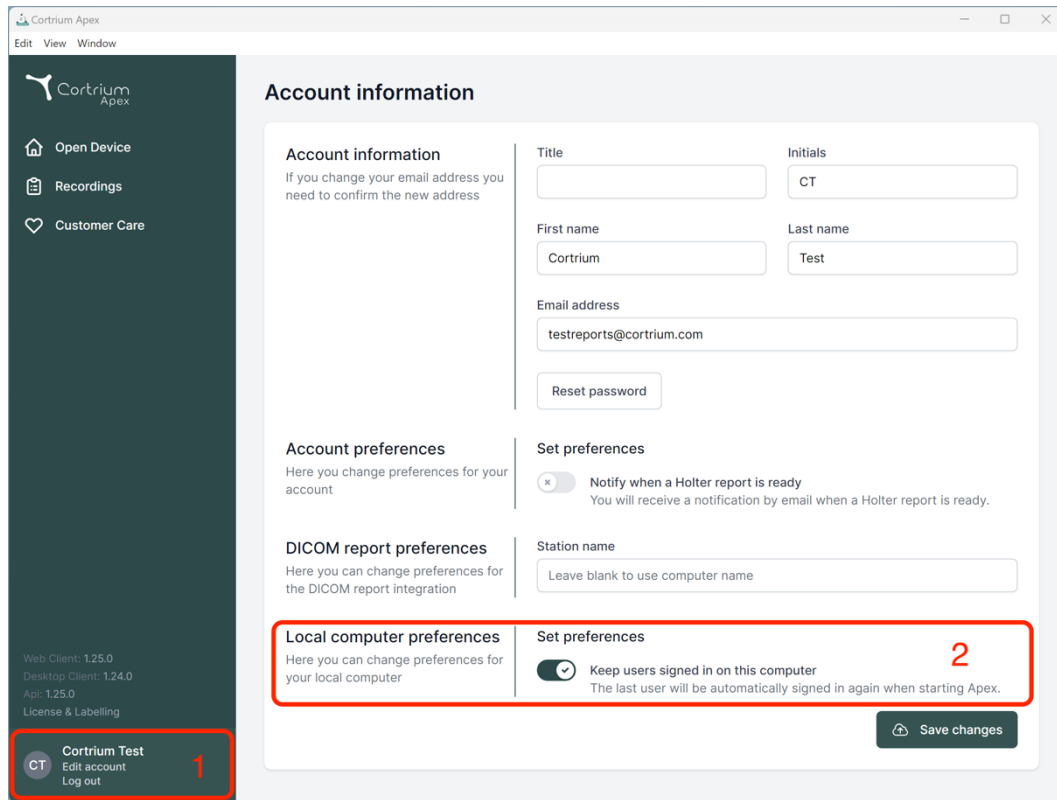The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0) and a 'License & Labelling' link. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section is highlighted with a red box and contains a 'Set preferences' toggle for 'Keep users signed in on this computer', which is currently checked and marked with a red '2'. A 'Save changes' button is located at the bottom right of the main content area.

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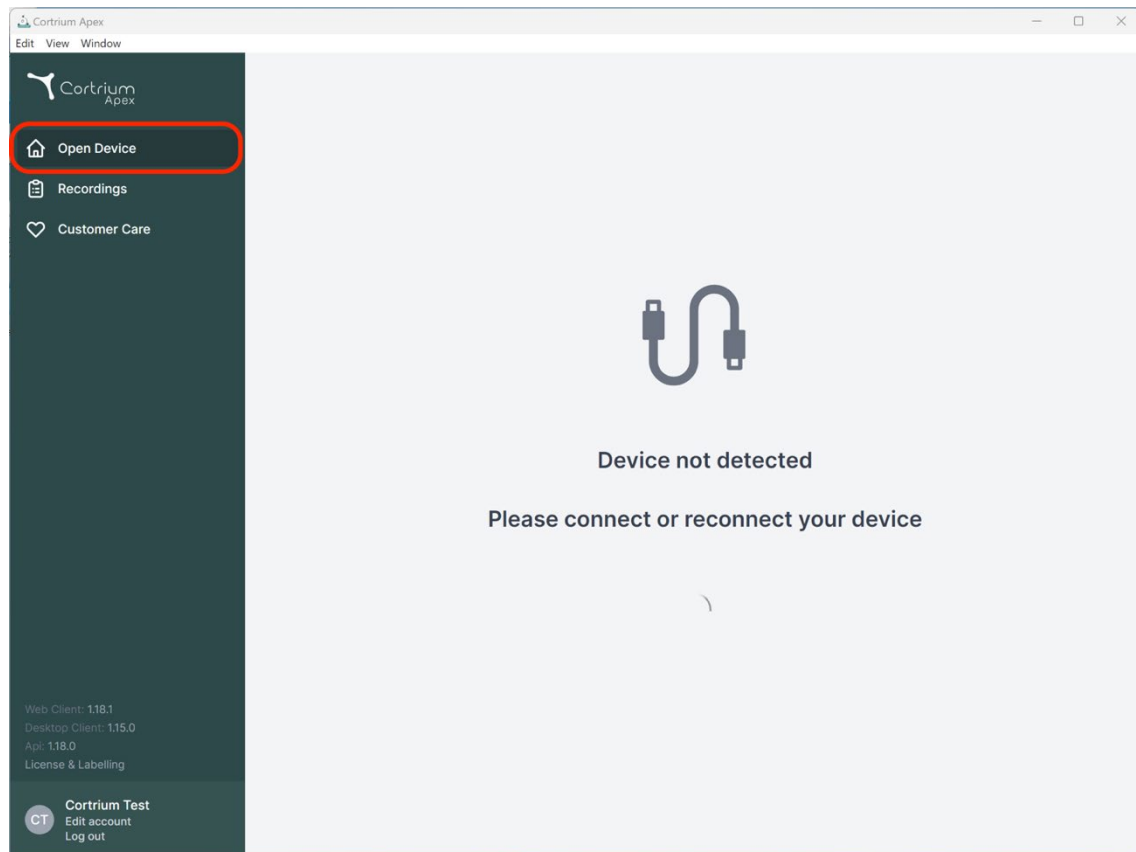
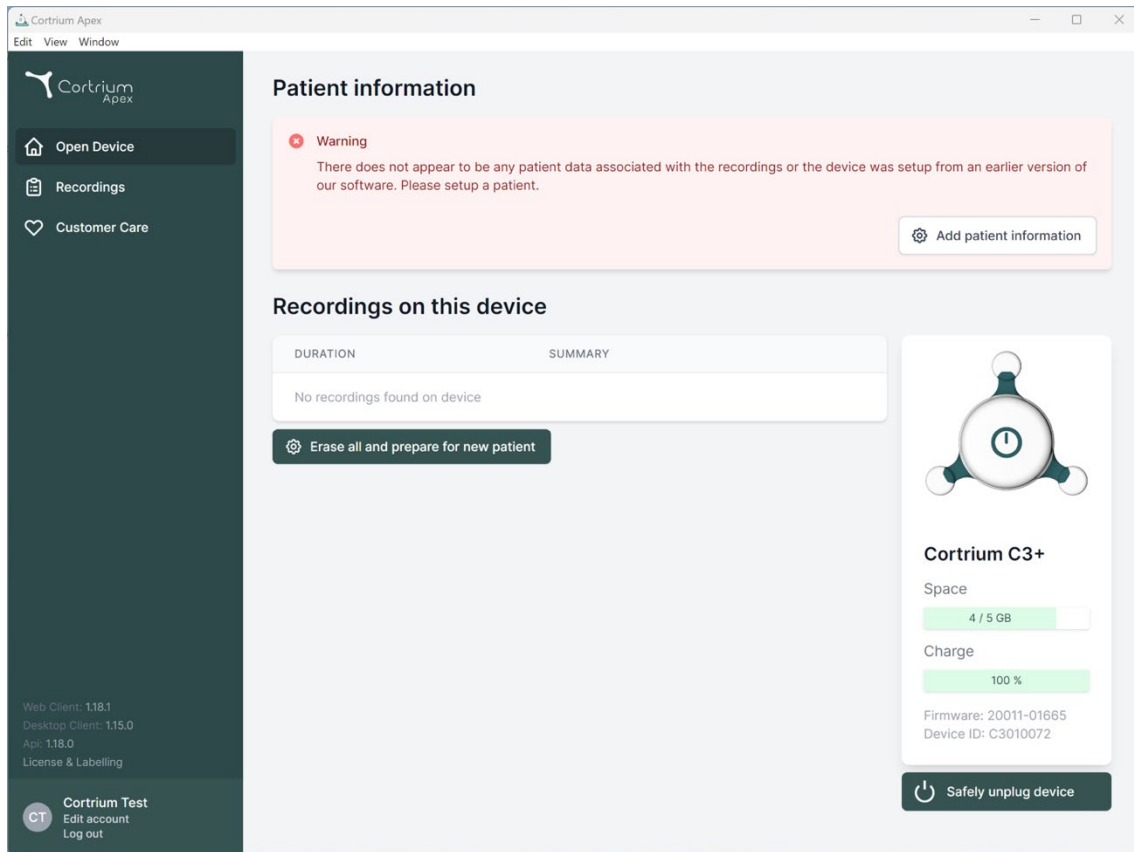
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning This will erase all recordings on the device

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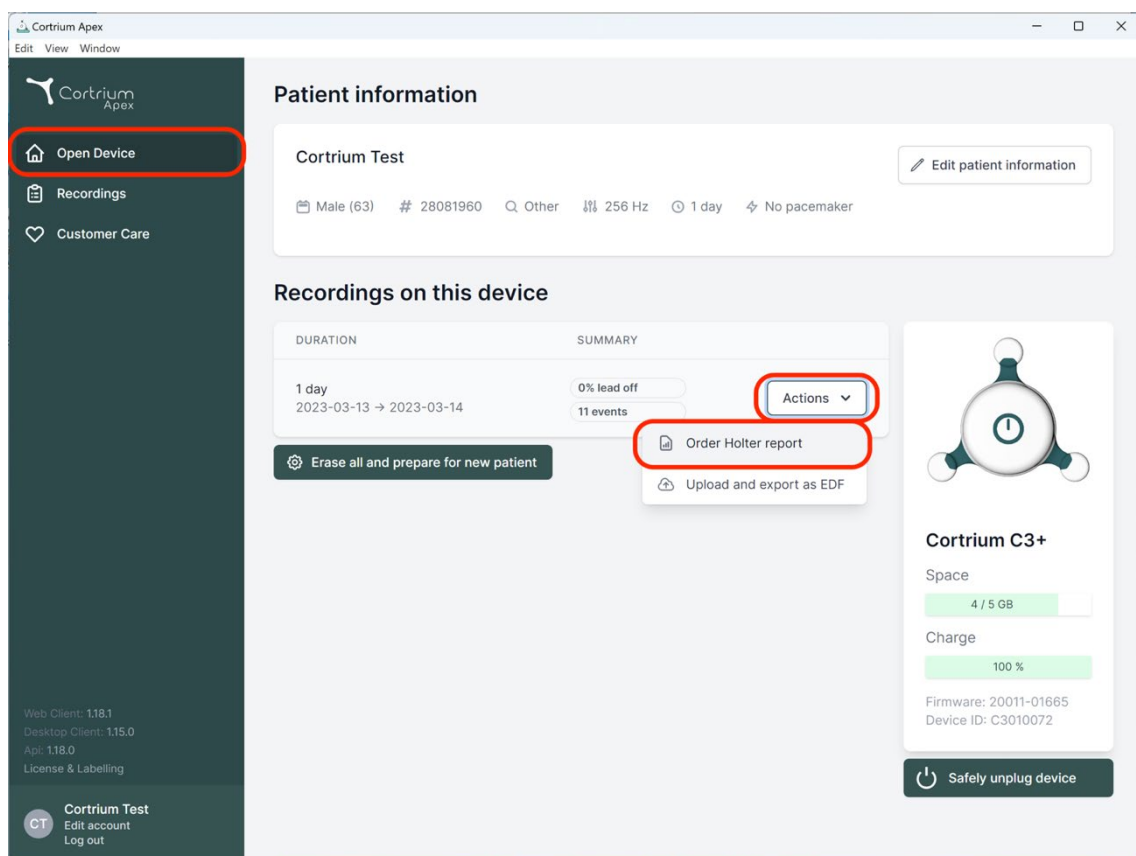
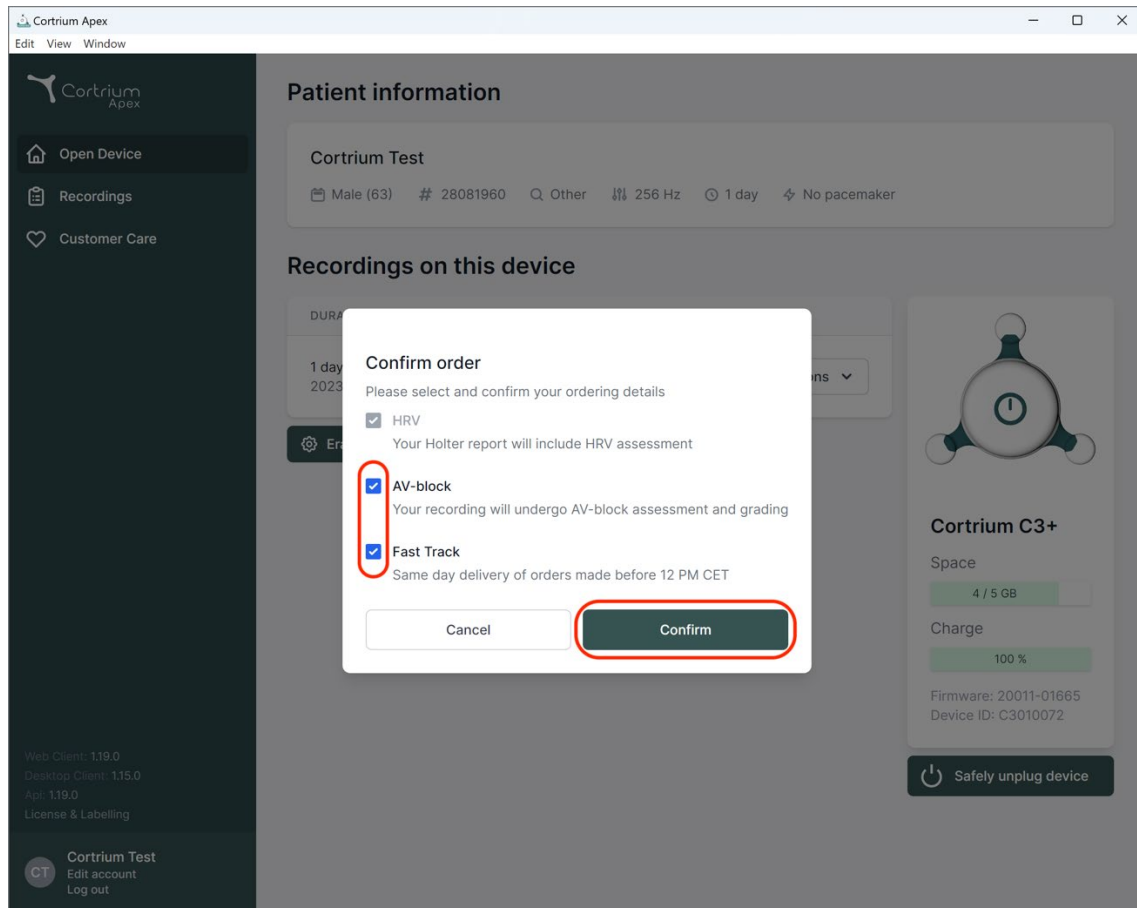
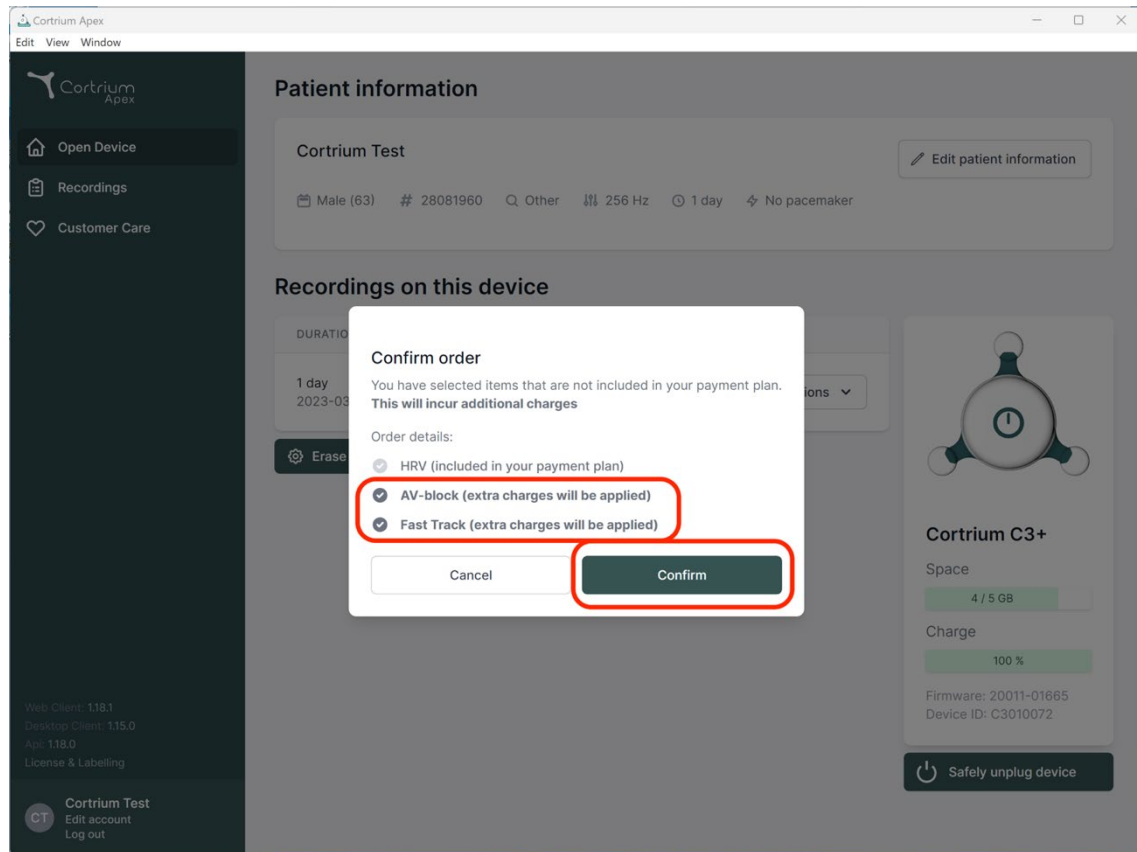
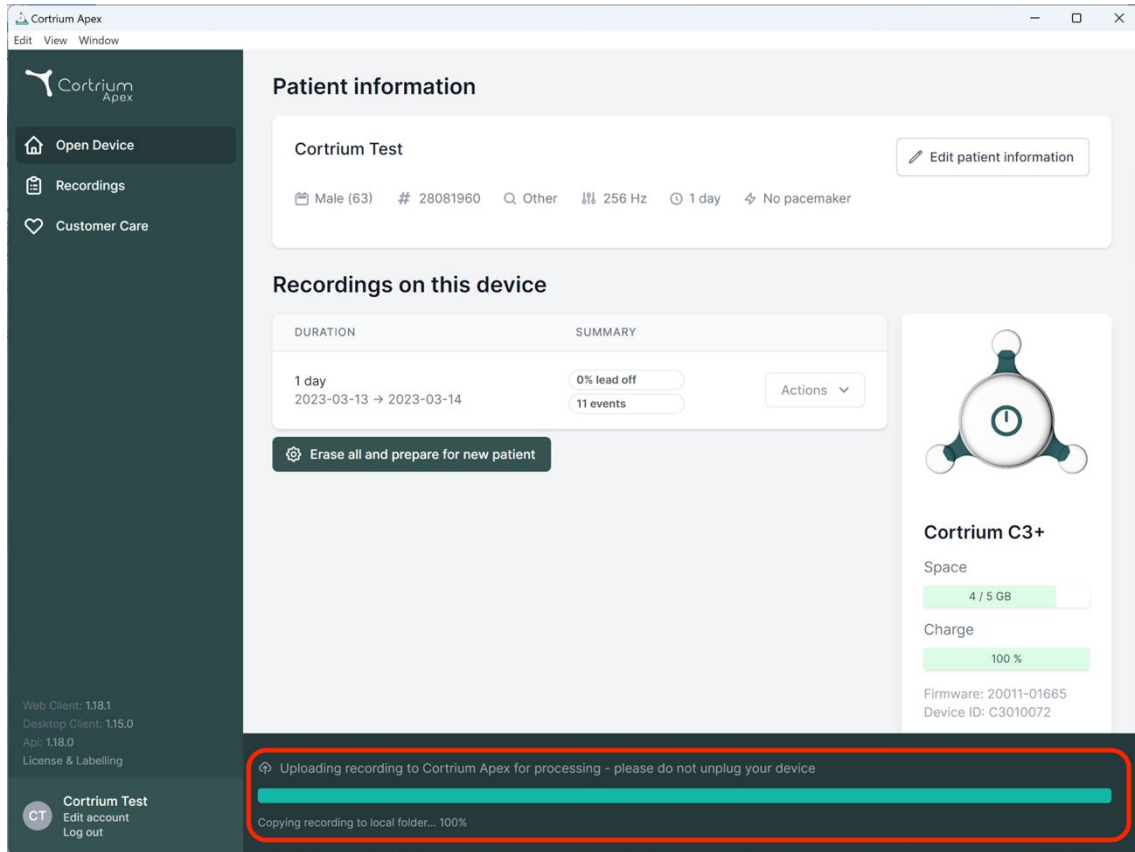
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and test parameters. Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom of the interface, which contains the text: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. To the right of the recording table, there is a device status card for 'Cortrium C3+' showing space usage (4 / 5 GB) and charge level (100%).

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
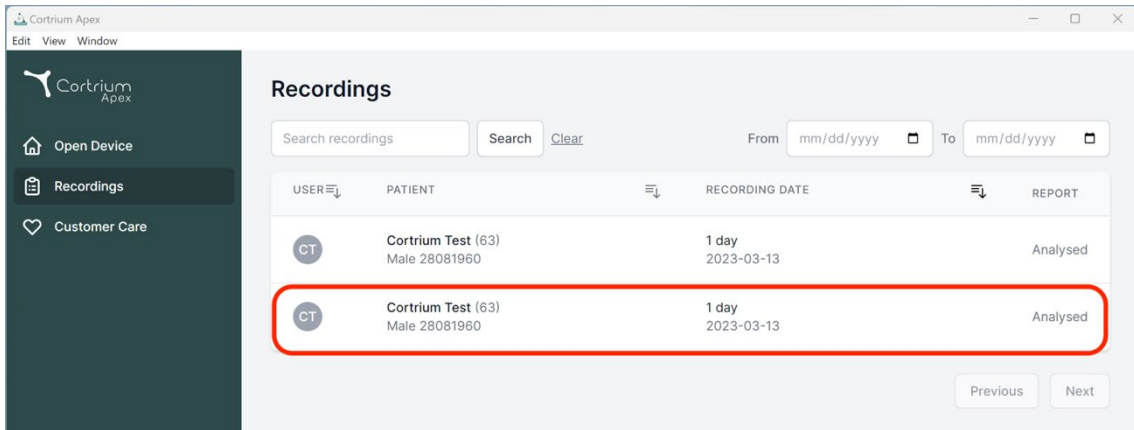
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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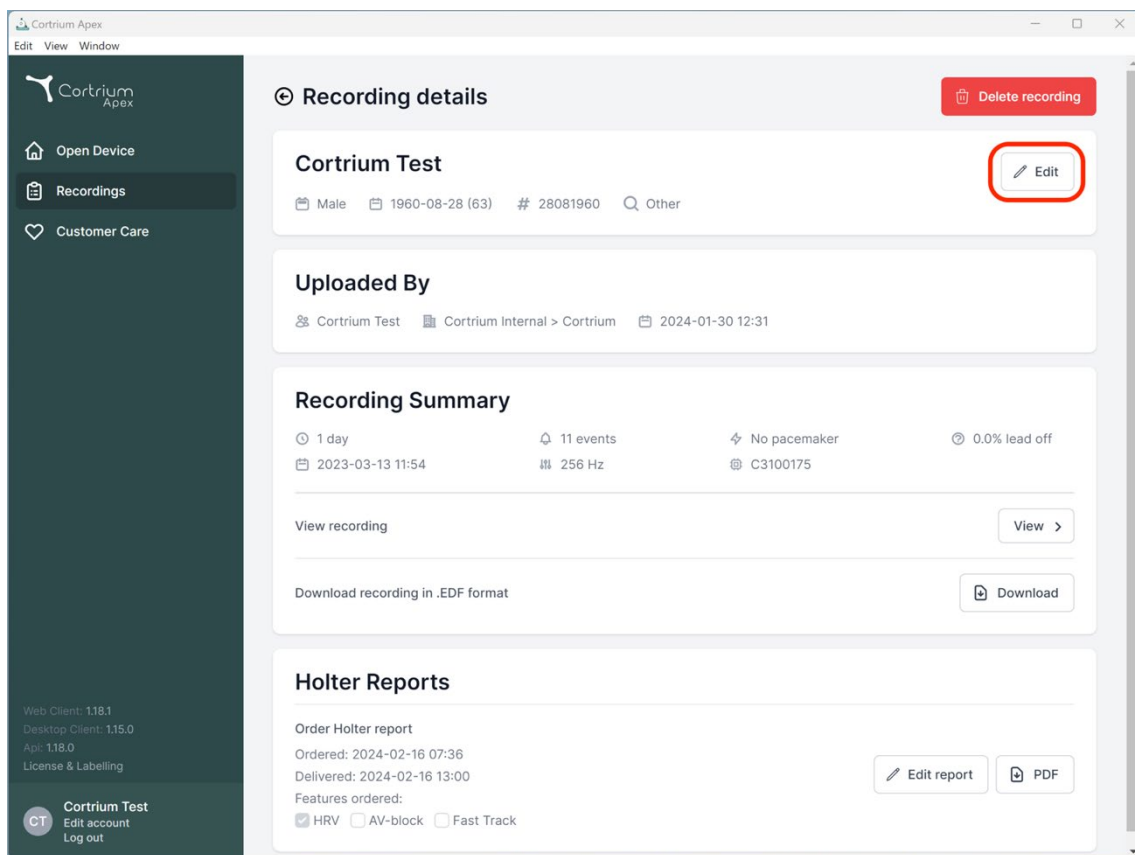2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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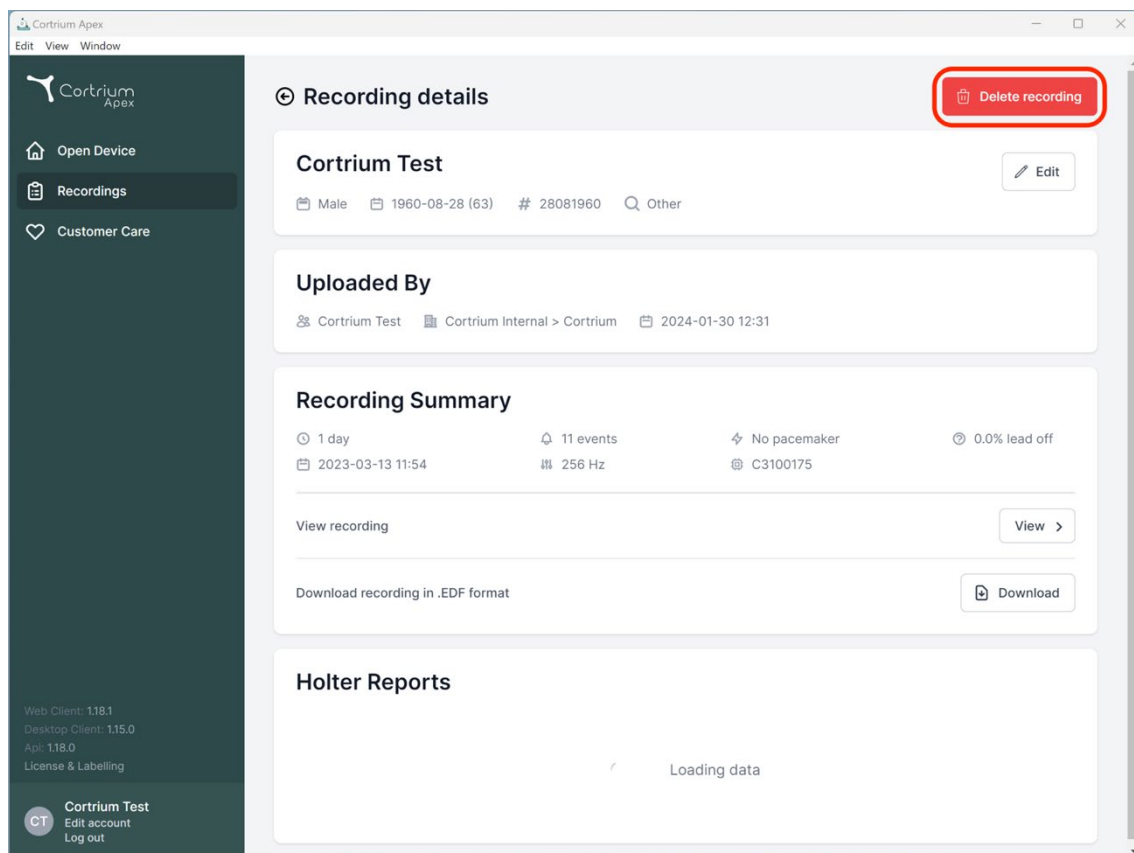
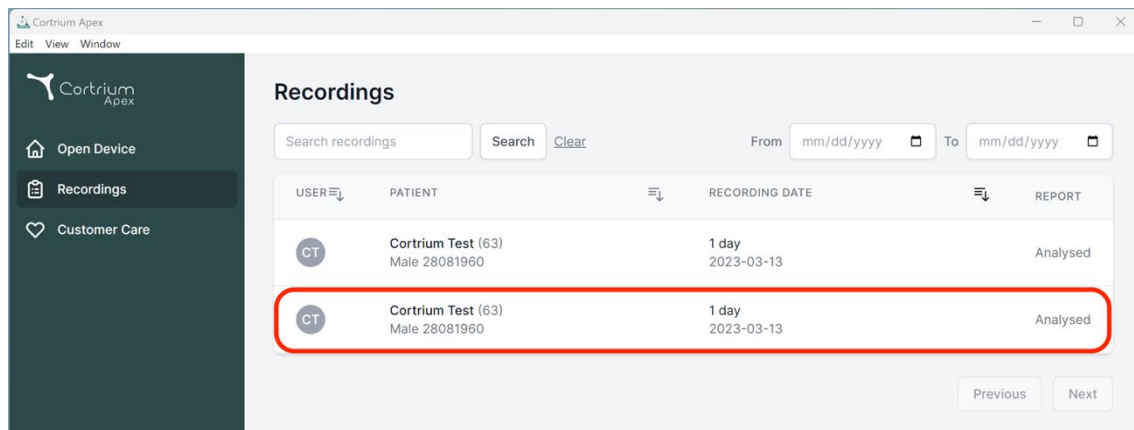
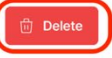
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31


Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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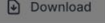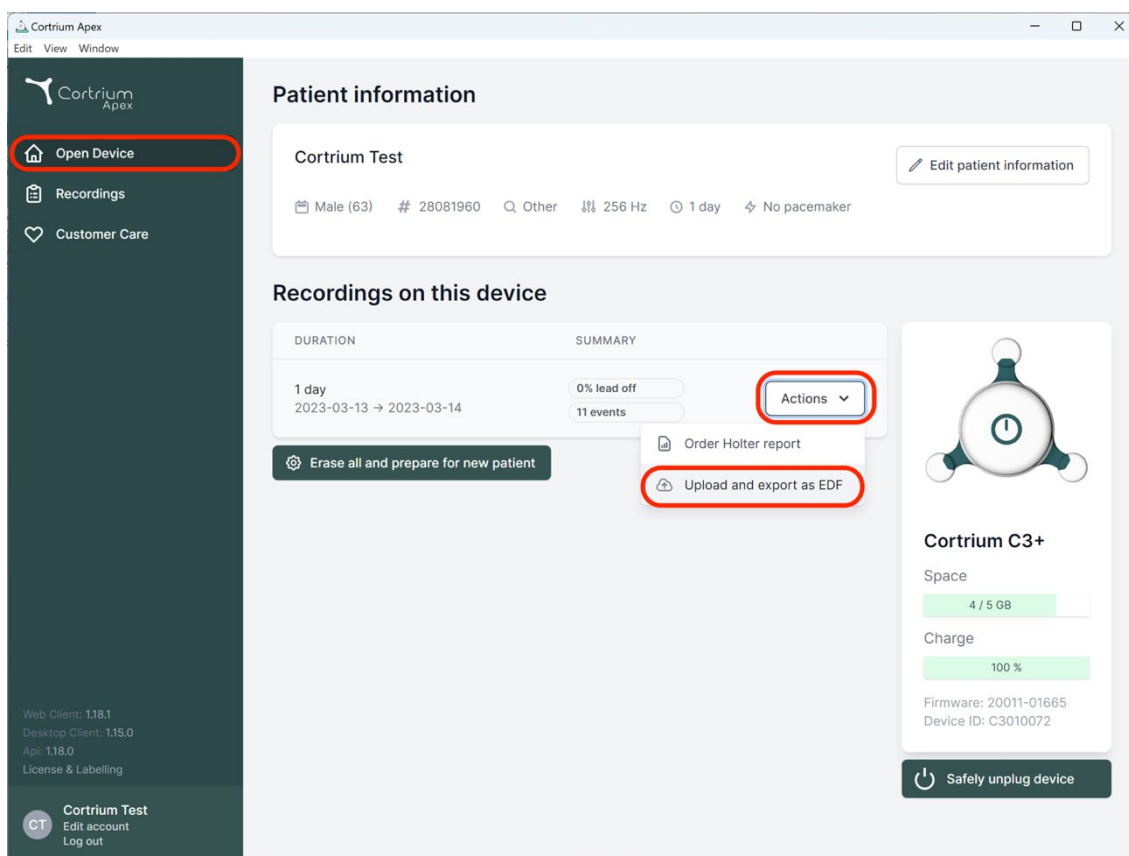
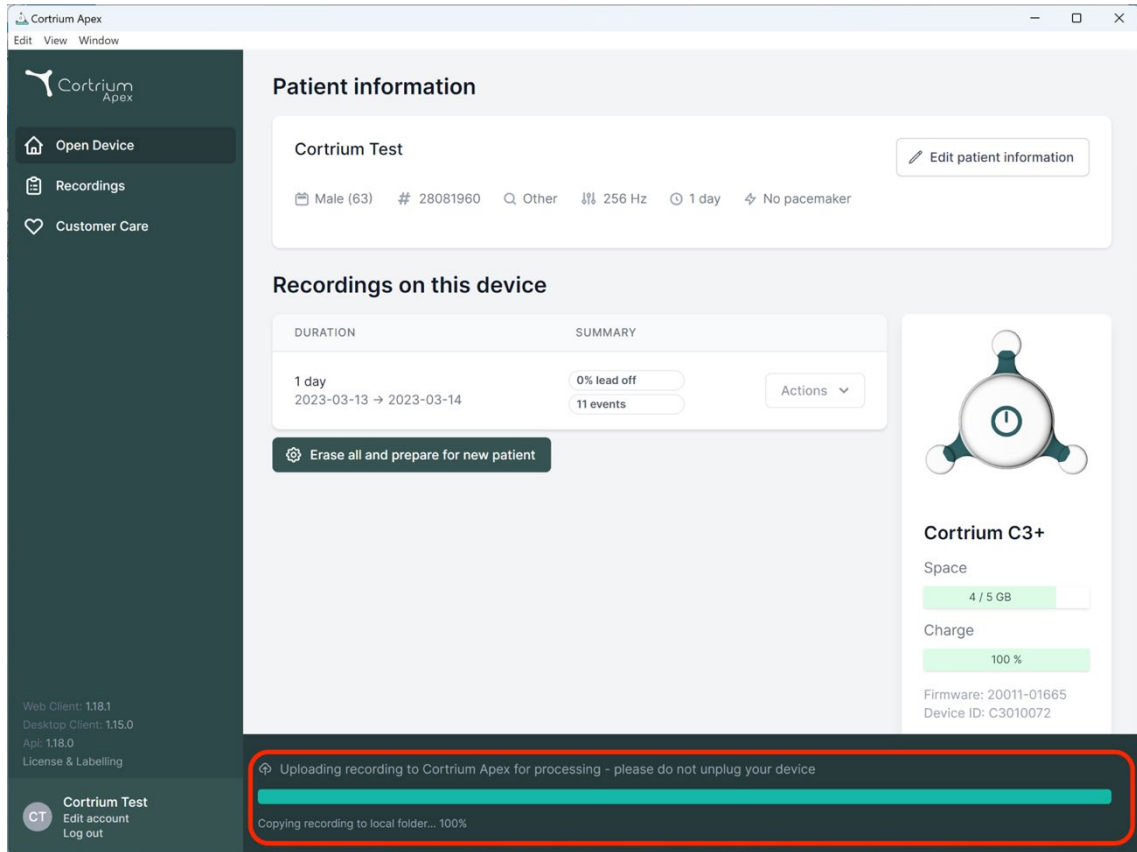
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

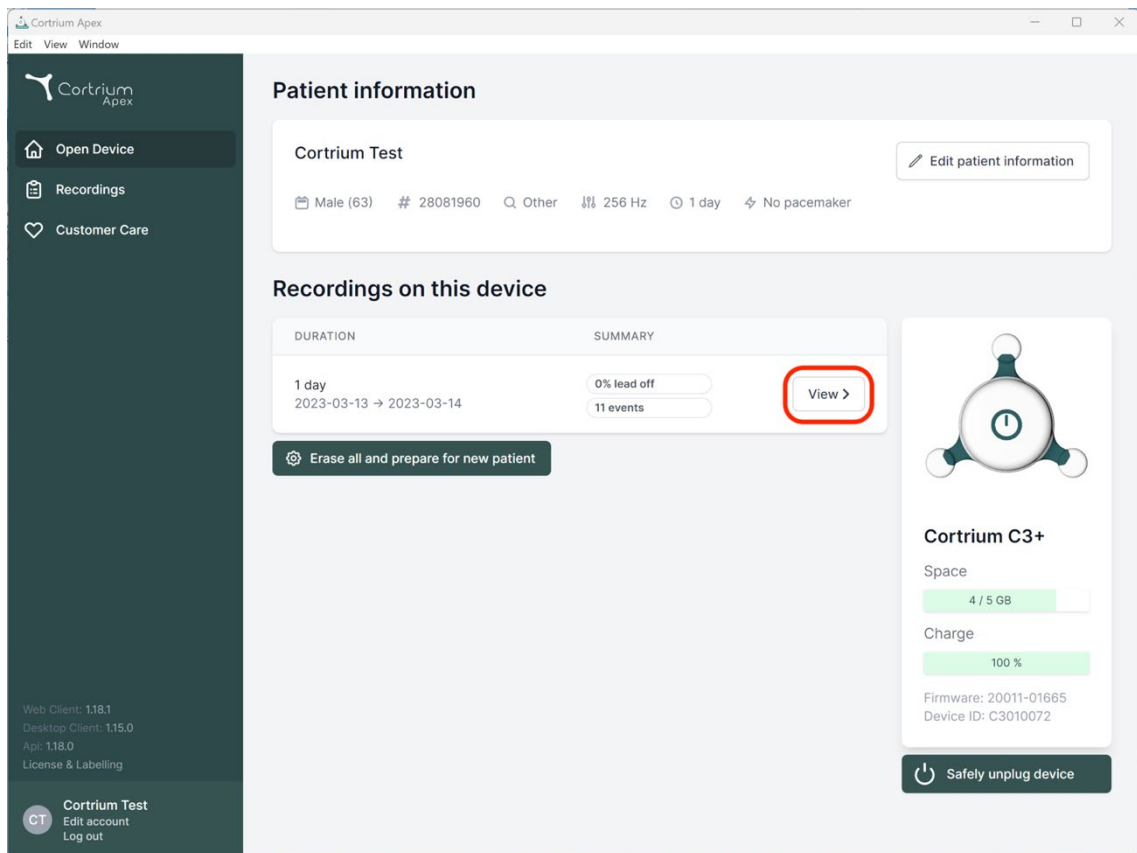
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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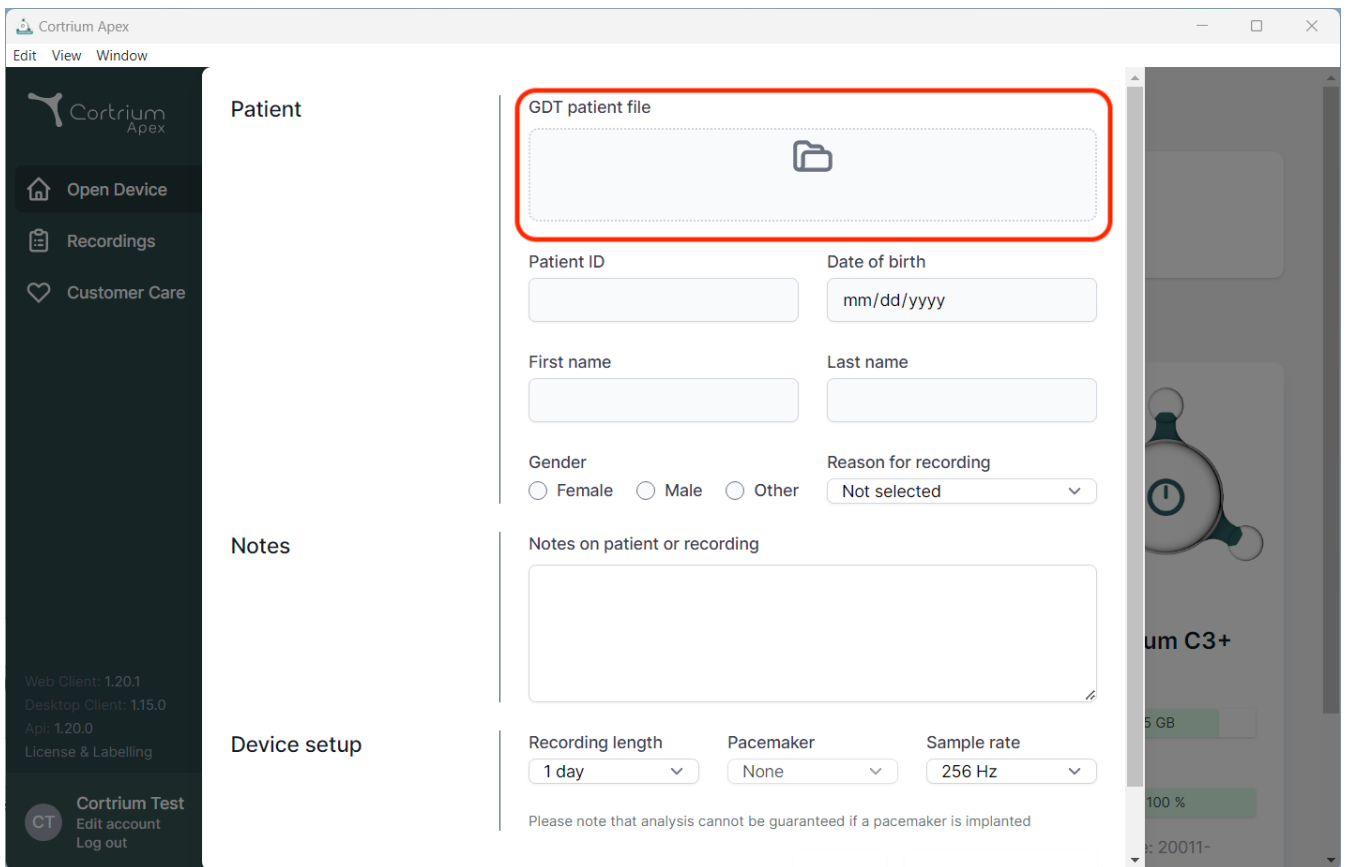
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

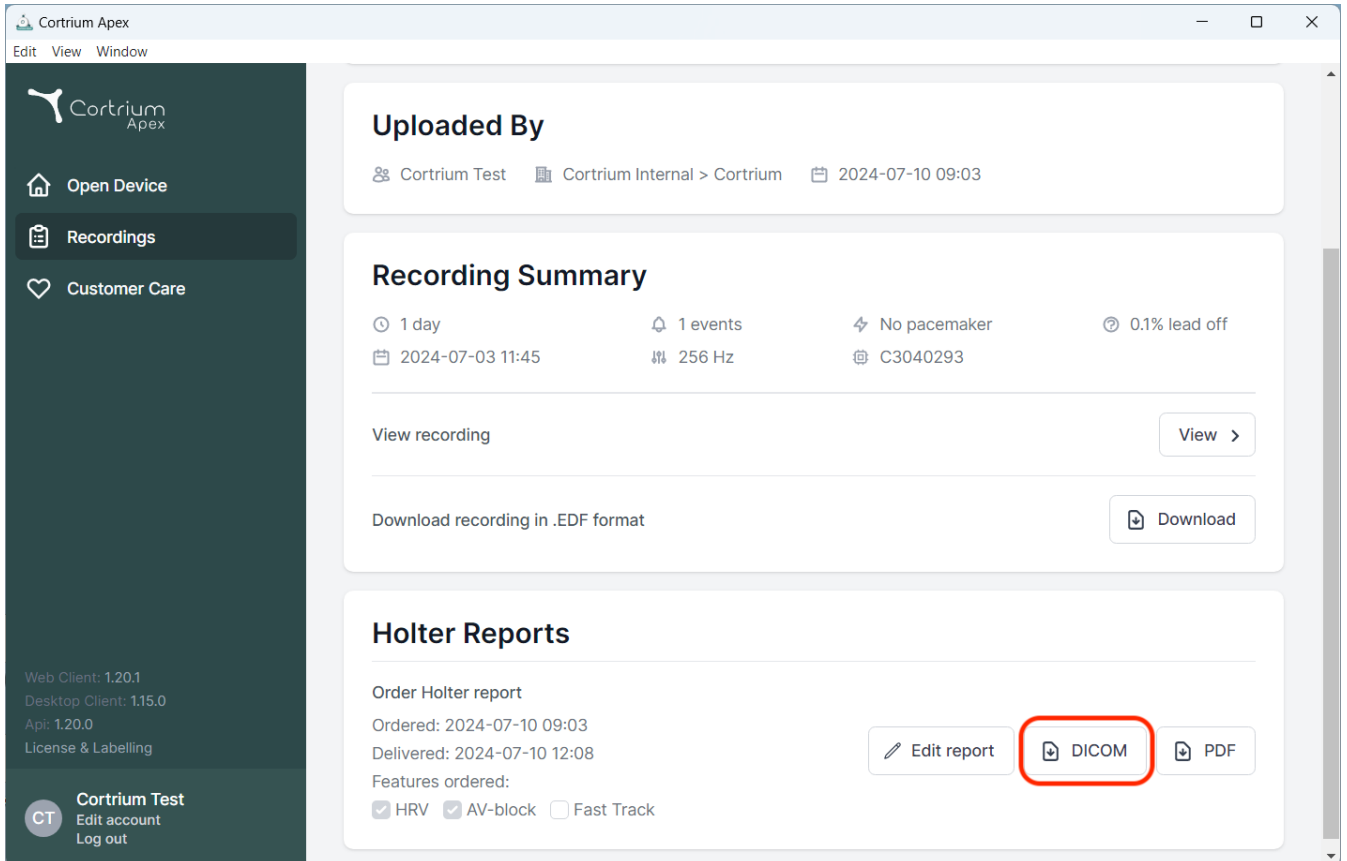
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section features a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a heart icon and the text "um C3+".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (for .EDF format).
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

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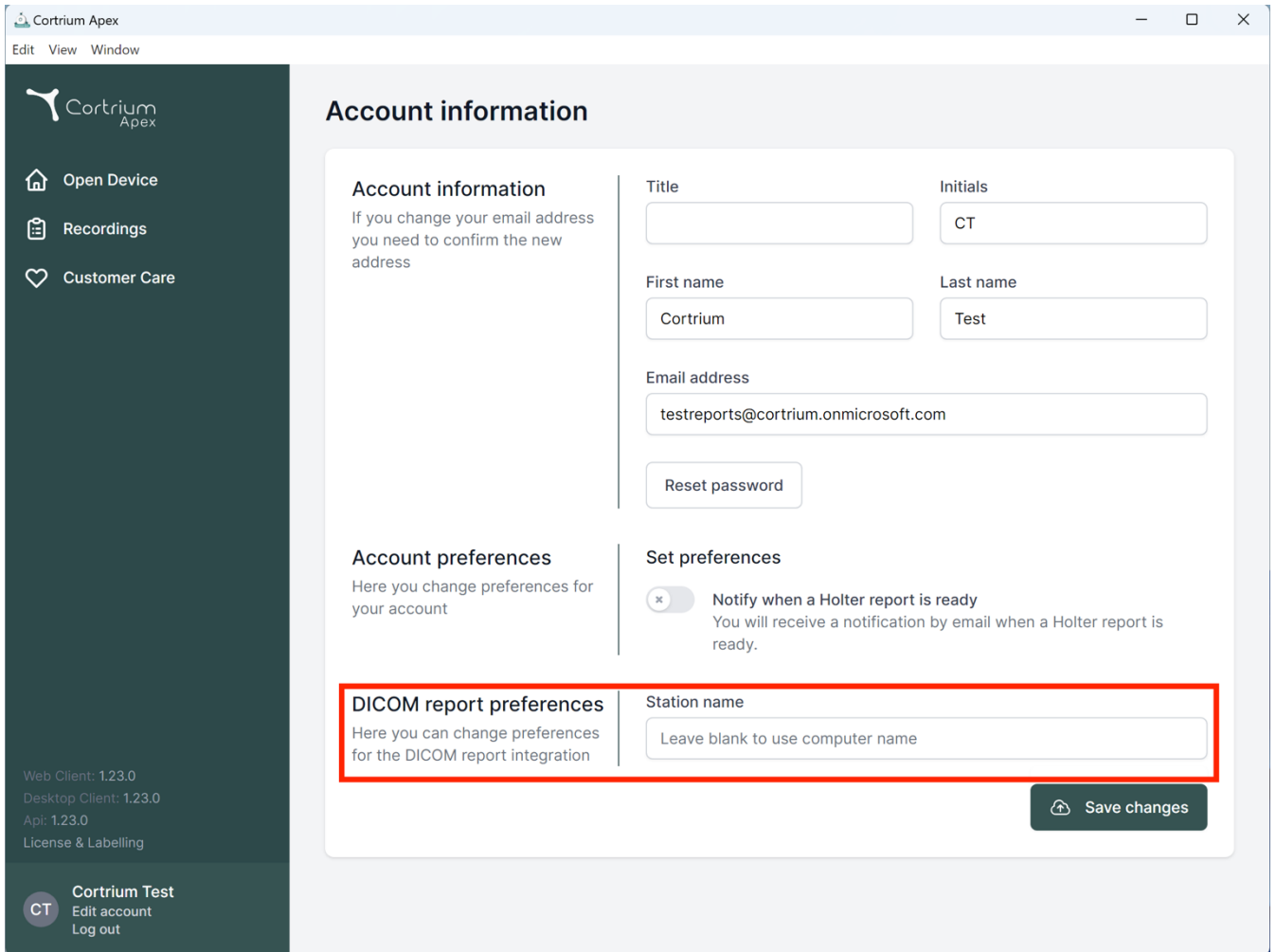Account information

If you change your email address you need to confirm the new address

Account information

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

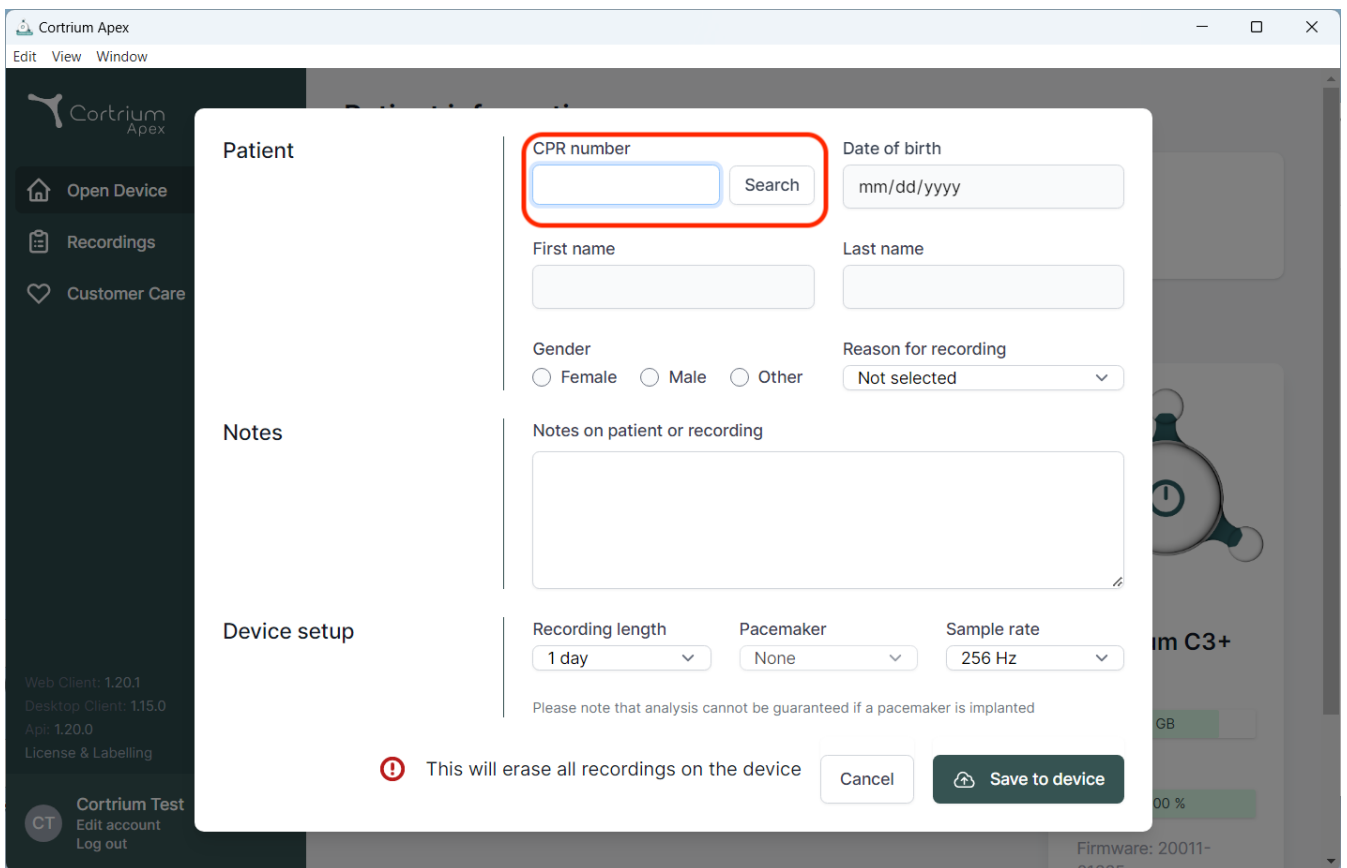
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot displays the Cortrium Apex web application interface. A modal form for patient information is open. The 'CPR number' field is highlighted with a red box. The form includes the following fields and options:

- Patient**
 - CPR number (highlighted with a red box) and a 'Search' button.
 - Date of birth (mm/dd/yyyy)
 - First name and Last name (text input fields).
 - Gender (Radio buttons: Female, Male, Other).
 - Reason for recording (Dropdown menu: Not selected).
- Notes**
 - Notes on patient or recording (Text area).
- Device setup**
 - Recording length (Dropdown menu: 1 day).
 - Pacemaker (Dropdown menu: None).
 - Sample rate (Dropdown menu: 256 Hz).

A warning message at the bottom states: "This will erase all recordings on the device". Below this message are 'Cancel' and 'Save to device' buttons.

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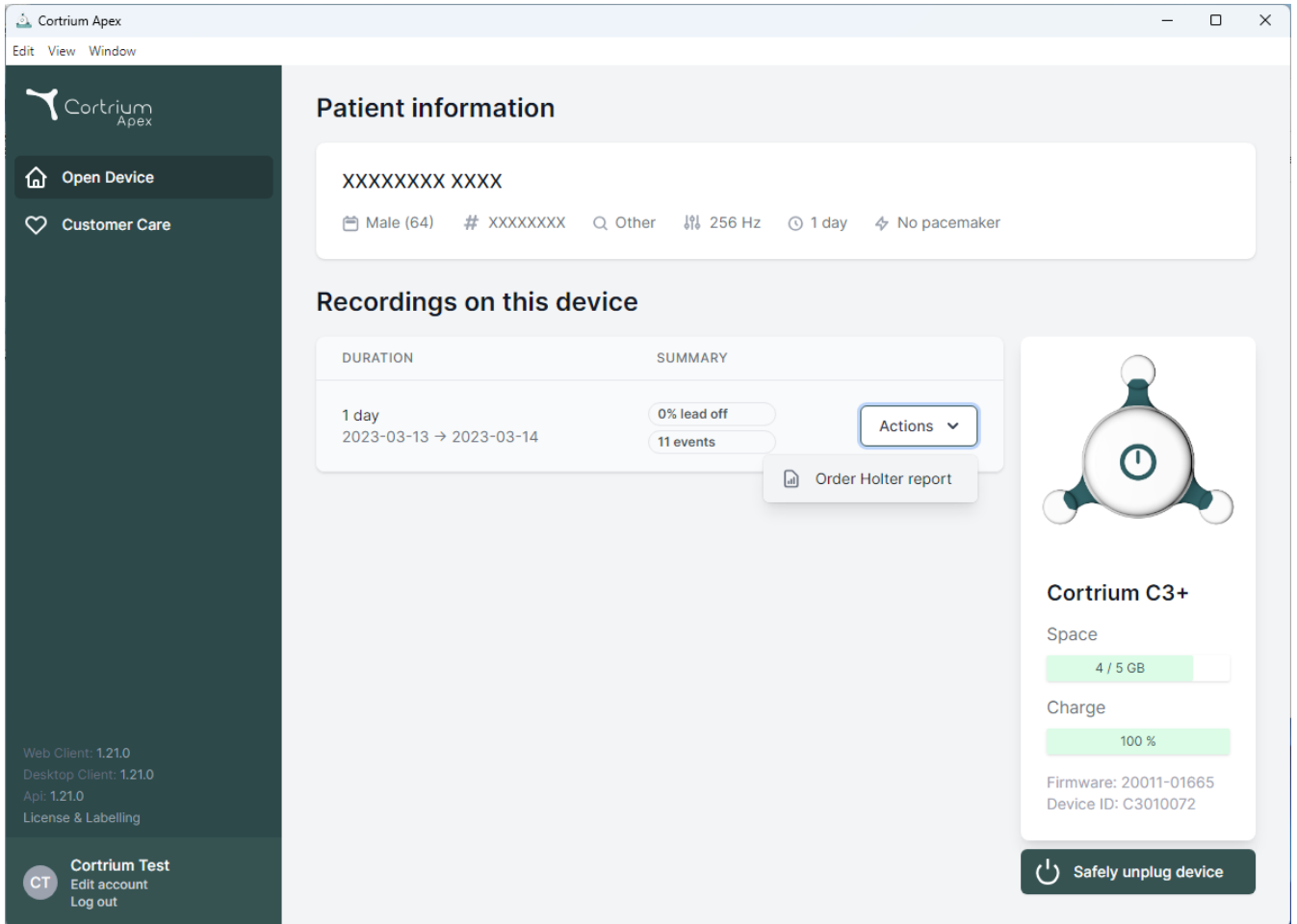
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), and 'Customer Care' (with a heart icon). Below these are version details: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar is a 'Cortrium Test' section with a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section contains a table with two columns: 'DURATION' and 'SUMMARY'. The table has one entry for '1 day' (2023-03-13 → 2023-03-14) with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space' (4 / 5 GB), 'Charge' (100 %), 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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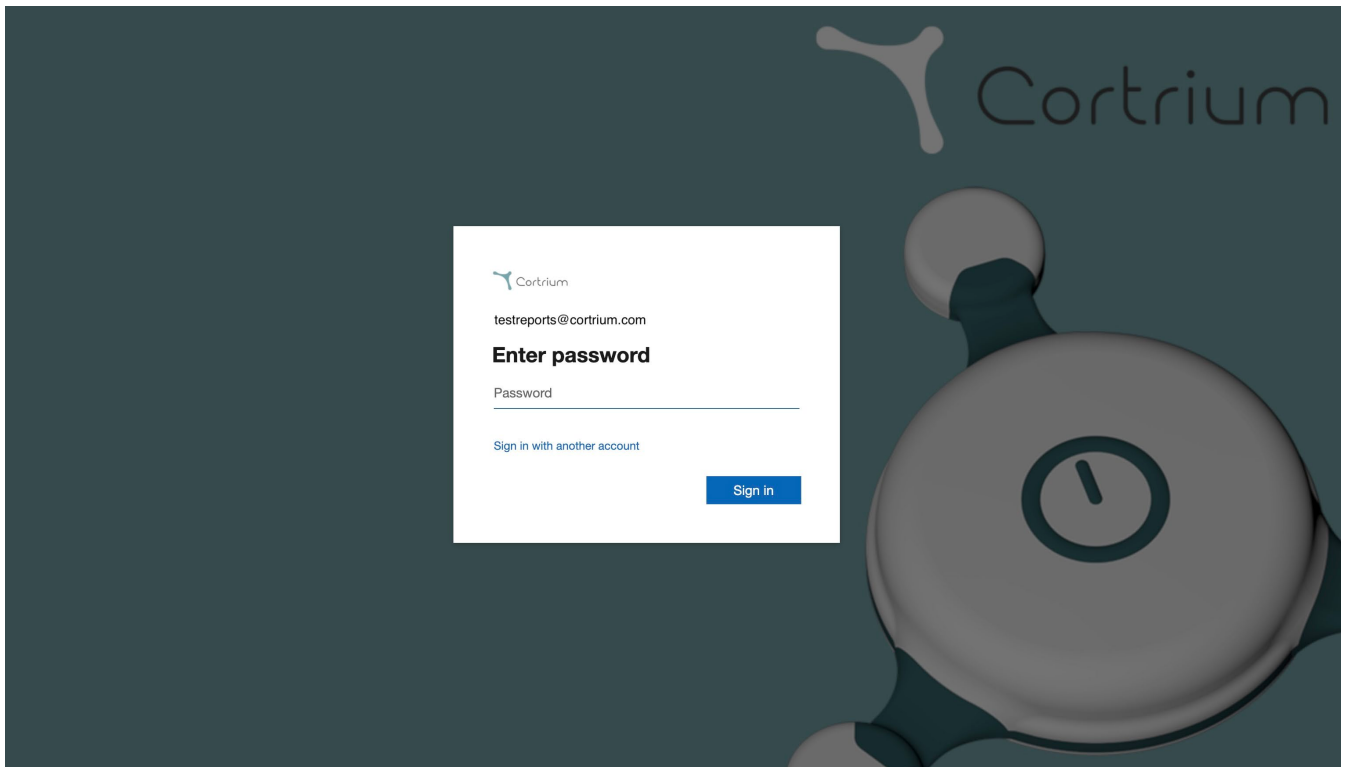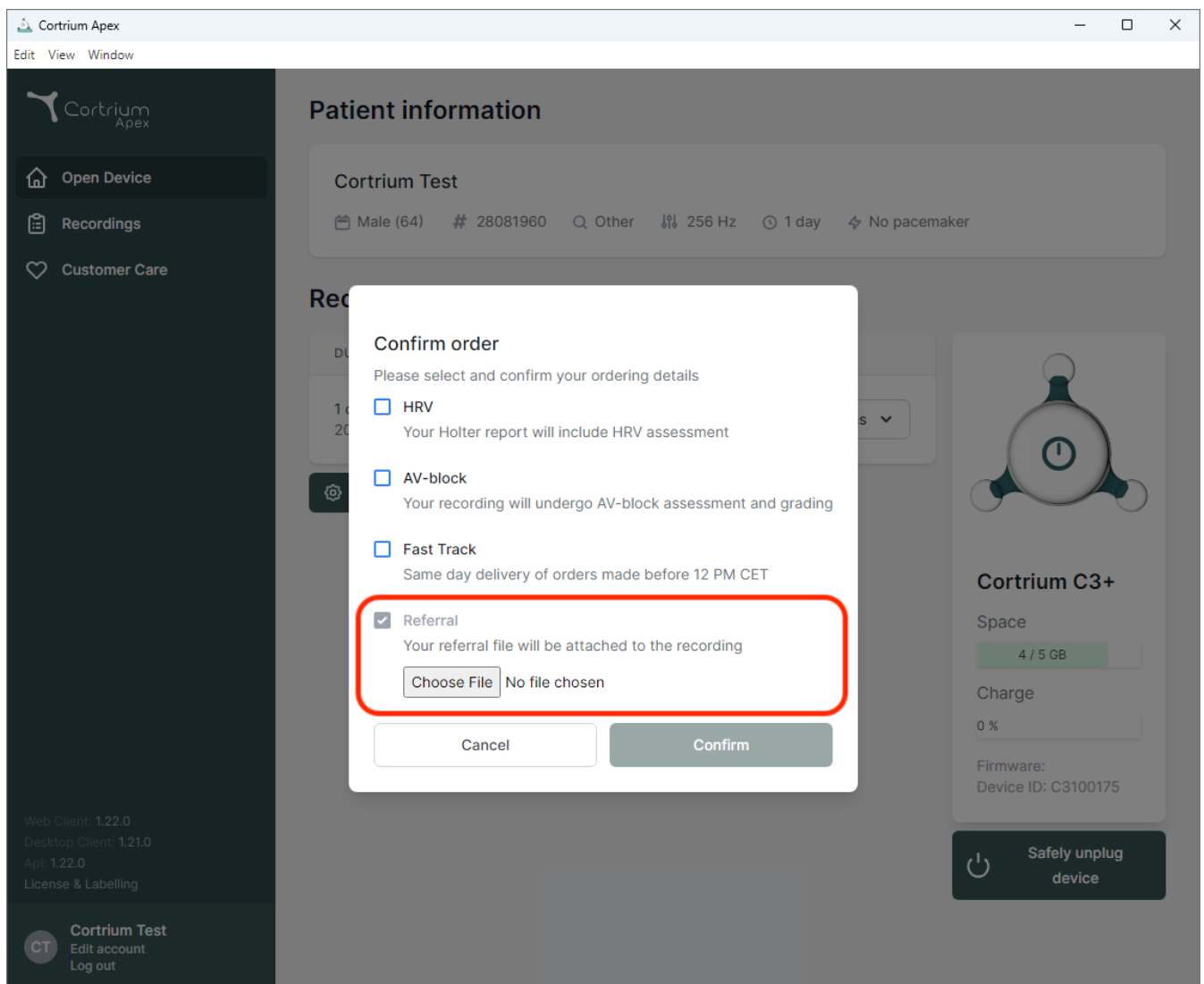
Test

User 03

Distributor 01 > Approver Organisation > Test Clinic

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
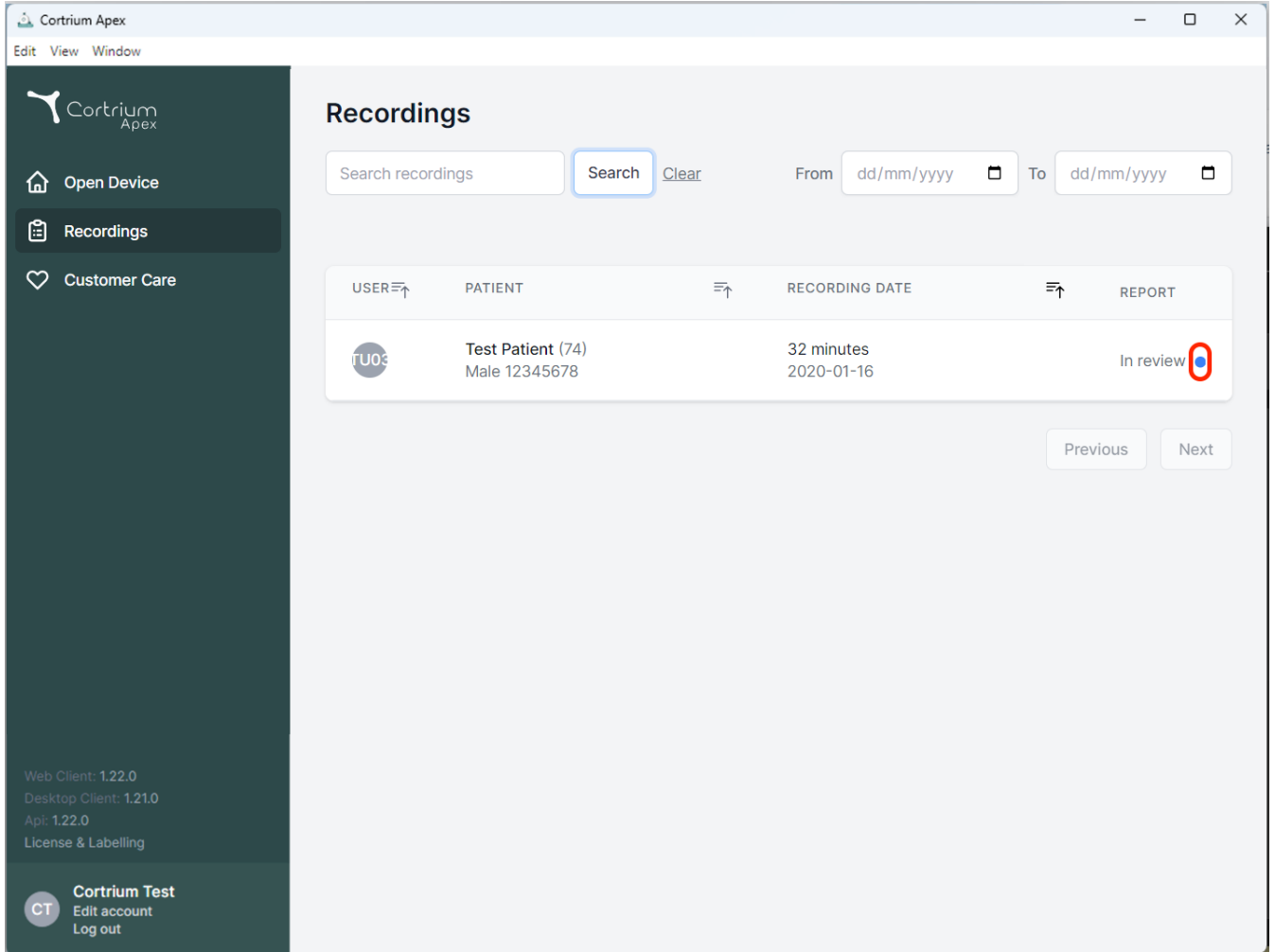
The screenshot shows the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area displays a table of recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for 'Test Patient (74)' with a duration of '32 minutes' and a date of '2020-01-16'. The 'REPORT' column shows 'In review' with a blue dot icon circled in red. Search and date filters are available at the top, and 'Previous' and 'Next' buttons are at the bottom right.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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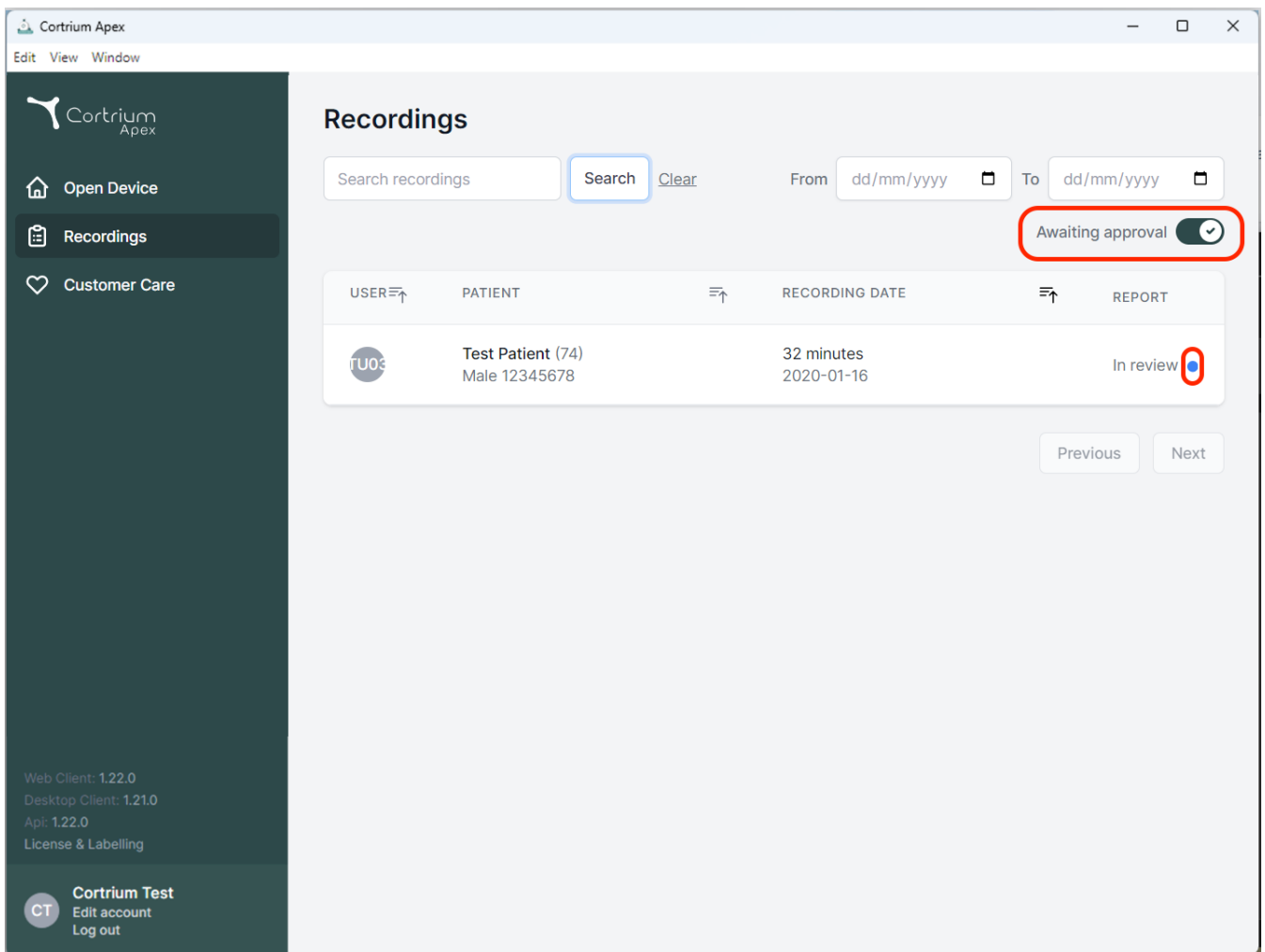
The screenshot displays the Cortrium Apex web application interface. The sidebar on the left includes the Cortrium Apex logo and navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area is titled "Recordings" and features a search bar, a "Search" button, a "Clear" link, and date range selectors for "From" and "To". A red box highlights the "Awaiting approval" toggle switch, which is currently turned on. Below this, a table lists recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. A red box highlights a blue dot in the "In review" status of a recording. The bottom of the sidebar shows version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling) and a "Cortrium Test" section with links for "Edit account" and "Log out".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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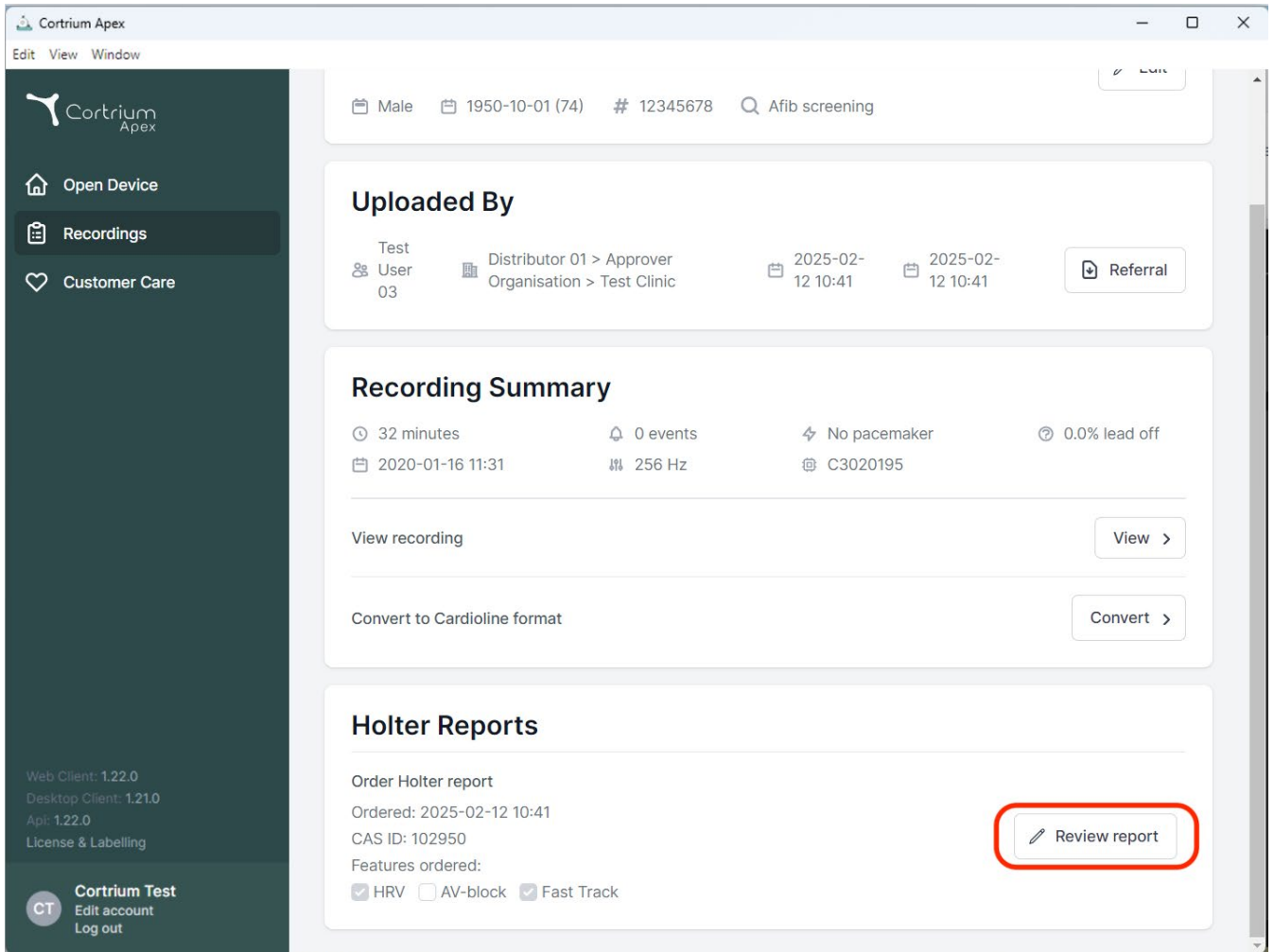The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41. A 'Referral' button is present. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It also shows '256 Hz' and 'C3020195'. Below the summary are buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report' with an order date of 2025-02-12 10:41, CAS ID 102950, and features ordered: HRV, AV-block, and Fast Track. A 'Review report' button is highlighted with a red rectangle.

Uploaded By

Test User 03
Distributor 01 > Approver Organisation > Test Clinic
2025-02-12 10:41
2025-02-12 10:41
Referral

Recording Summary

32 minutes
0 events
No pacemaker
0.0% lead off
2020-01-16 11:31
256 Hz
C3020195

View recording
Convert to Cardioline format

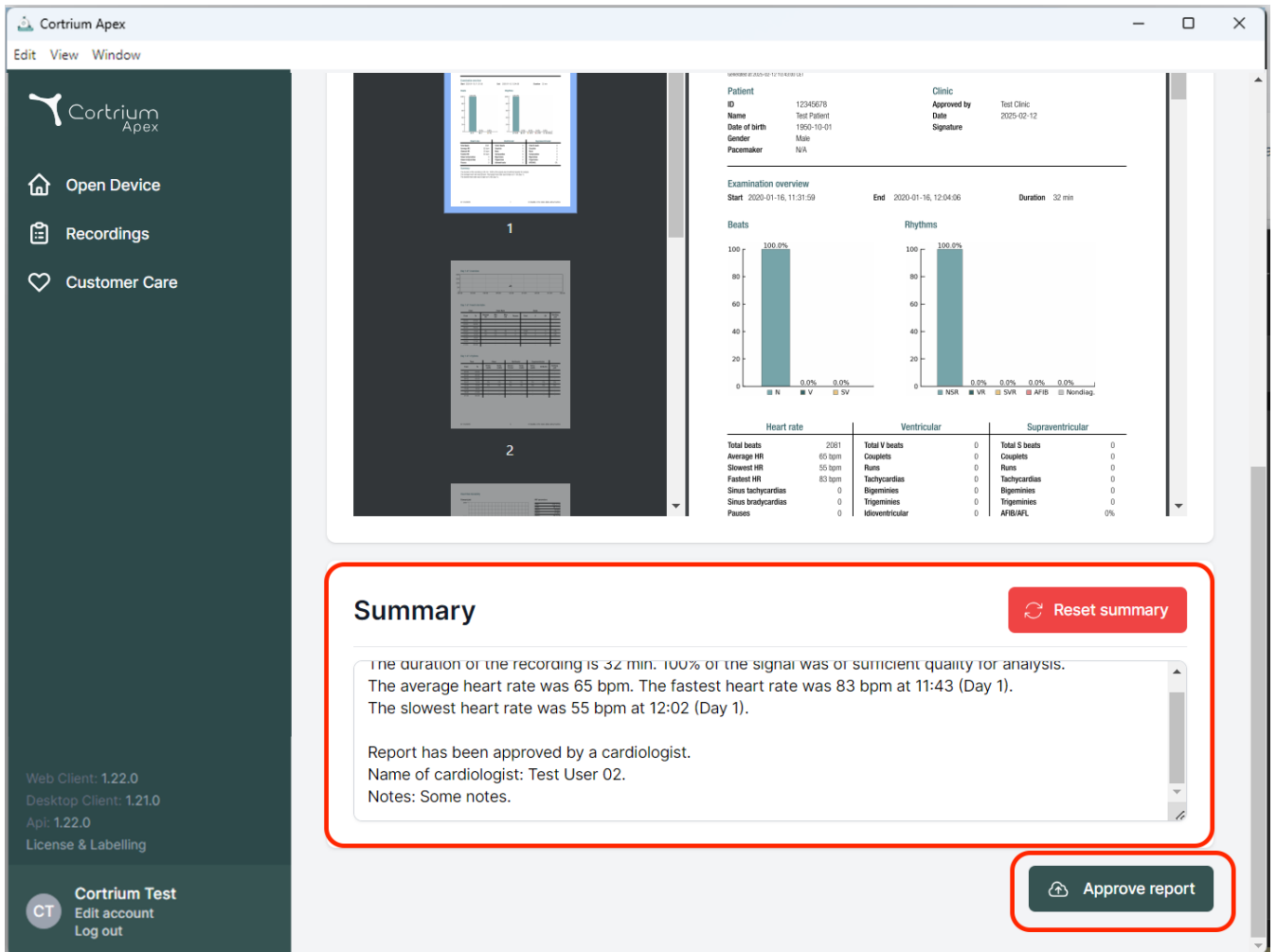
Holter Reports

Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track
Review report

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Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

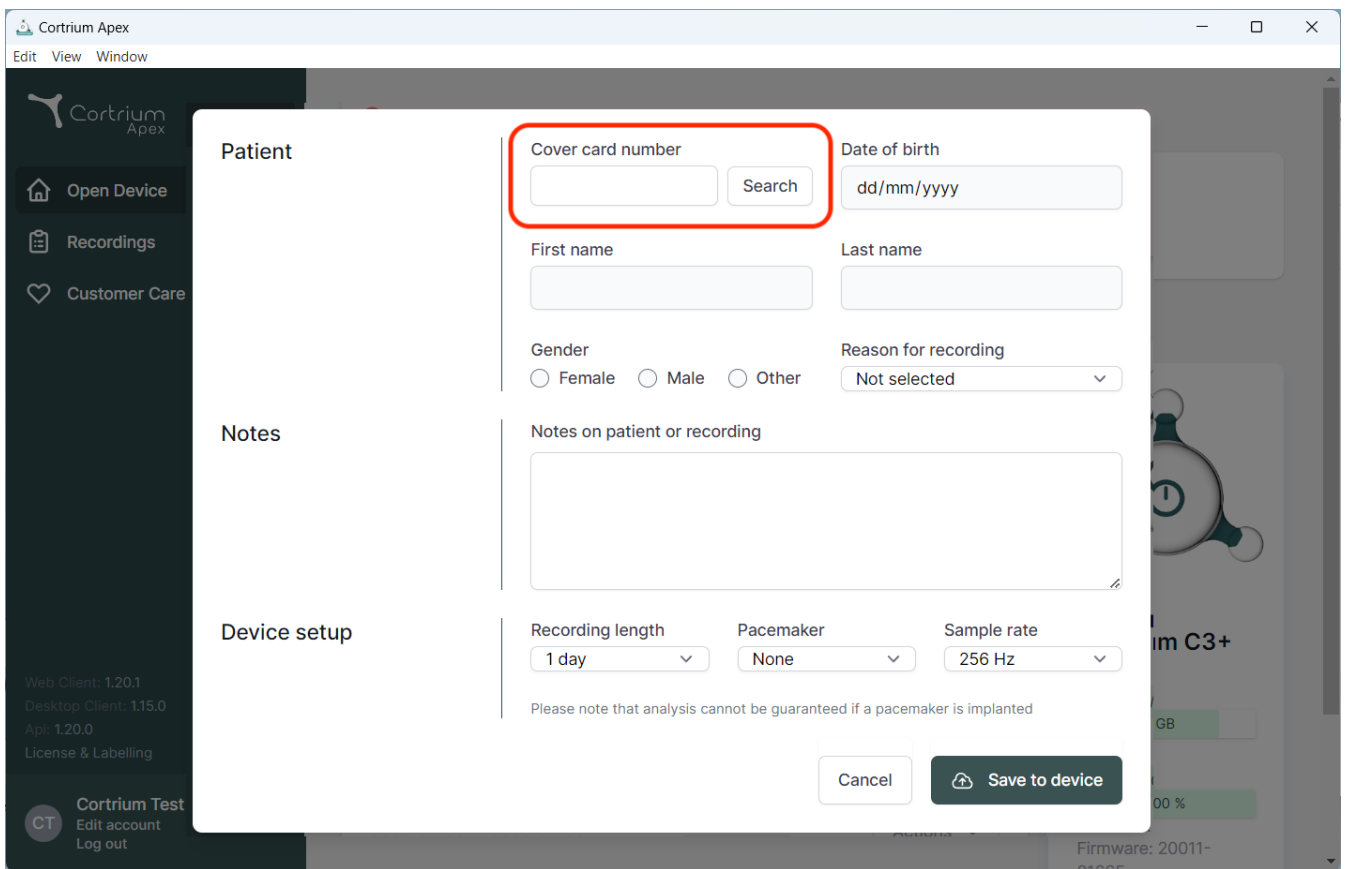
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient search and device setup. The form includes fields for "Cover card number" (highlighted with a red box), "Date of birth", "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording", and "Notes on patient or recording". Below these fields are "Recording length", "Pacemaker", and "Sample rate" dropdowns. A "Cancel" button and a "Save to device" button are at the bottom right. The background shows the main application menu with options like "Open Device", "Recordings", and "Customer Care".

Patient

Cover card number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47