Cortrium Apex 1.26

Cortrium Apex 1.26

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
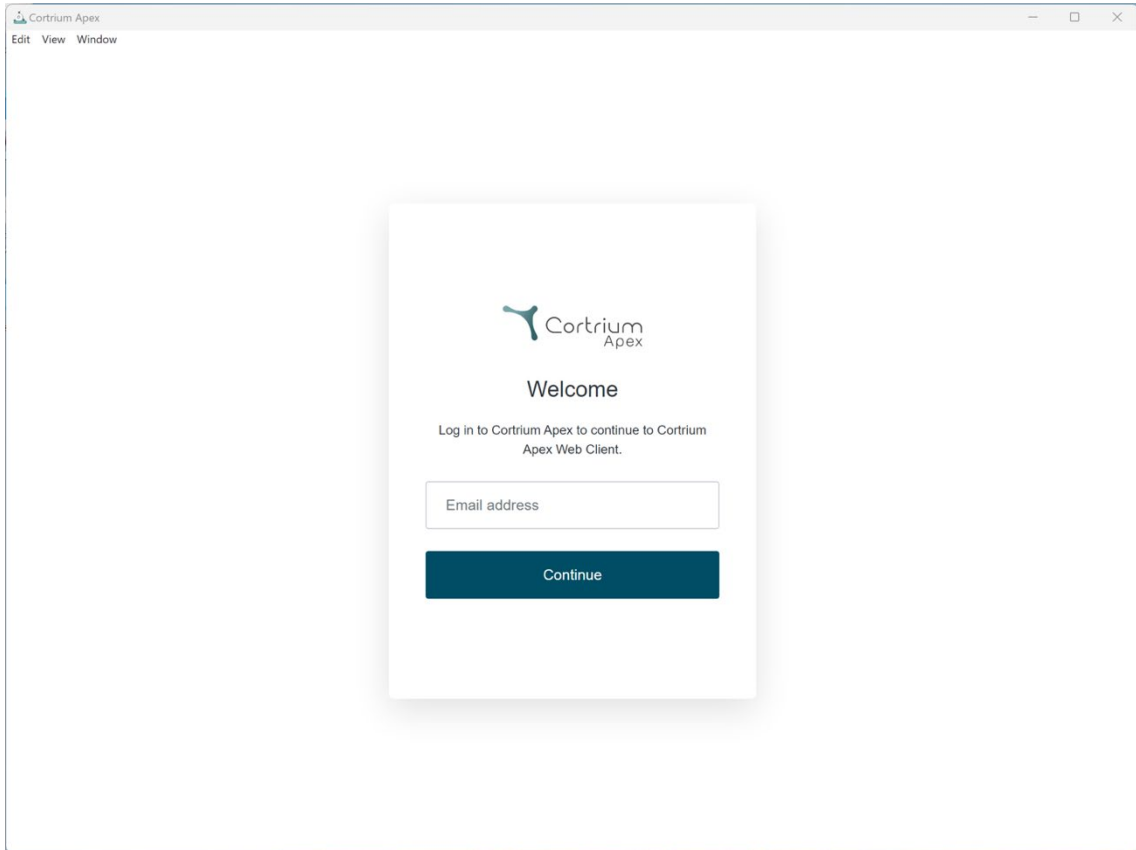
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". In the center of the browser is a white login dialog box with a shadow. The dialog box features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" at the bottom of the dialog.

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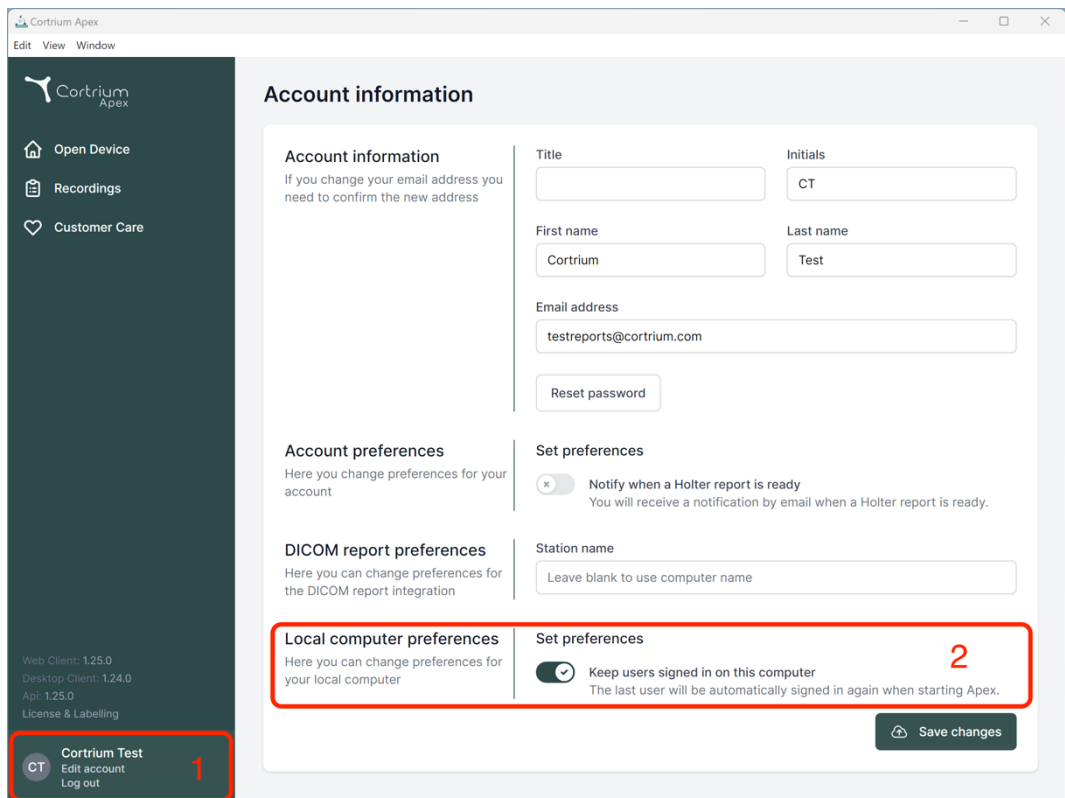
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.25.0', 'Desktop Client: 1.24.0', 'Api: 1.25.0', and 'License & Labelling'. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section includes a 'Set preferences' area with a toggle for 'Keep users signed in on this computer', which is currently turned on and marked with a red '2'. A 'Save changes' button is located at the bottom right of the main content area.

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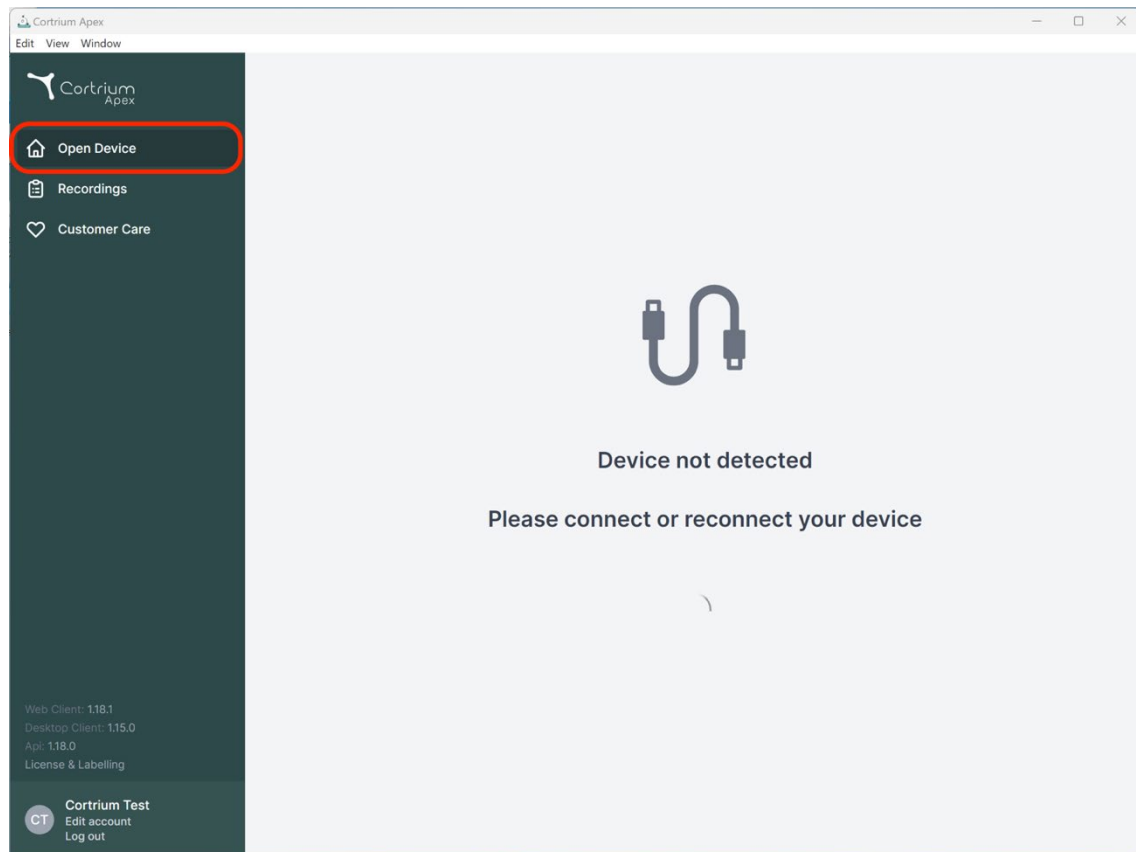
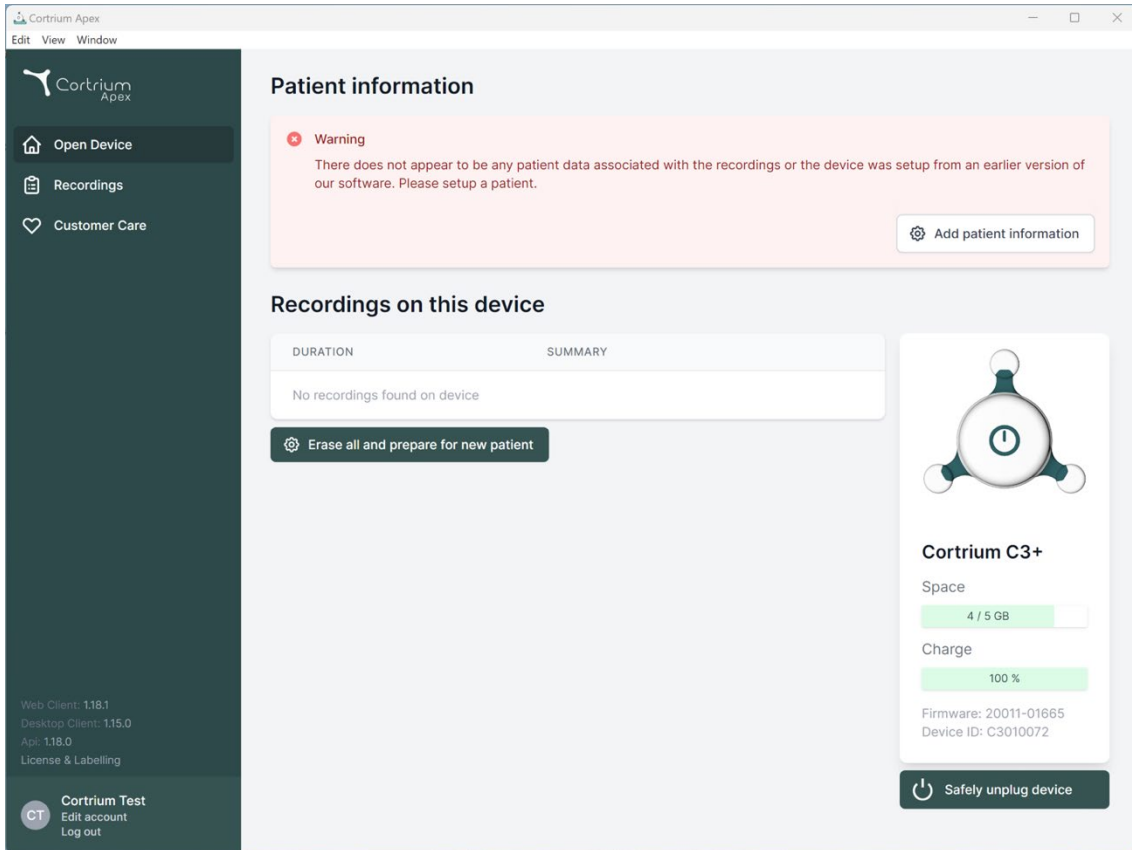
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, it lists version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. Below this is a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

The main content area has a light gray background. At the top, under the heading 'Patient information', there is a pink warning box with a red 'x' icon. The warning text states: 'There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' To the right of the warning box is a button labeled 'Add patient information' with a gear icon.

Below the warning box is the section 'Recordings on this device'. It contains a table with two columns: 'DURATION' and 'SUMMARY'. The table is currently empty, with the text 'No recordings found on device' centered below the headers. Below the table is a dark button labeled 'Erase all and prepare for new patient' with a gear icon.

On the right side of the main content area is a white card for 'Cortrium C3+'. It features a circular icon with a clock face. Below the icon, it shows 'Space' usage as '4 / 5 GB' with a green progress bar, and 'Charge' status as '100 %' with a green progress bar. At the bottom of the card, it lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. Below the card is a dark button labeled 'Safely unplug device' with a power icon.

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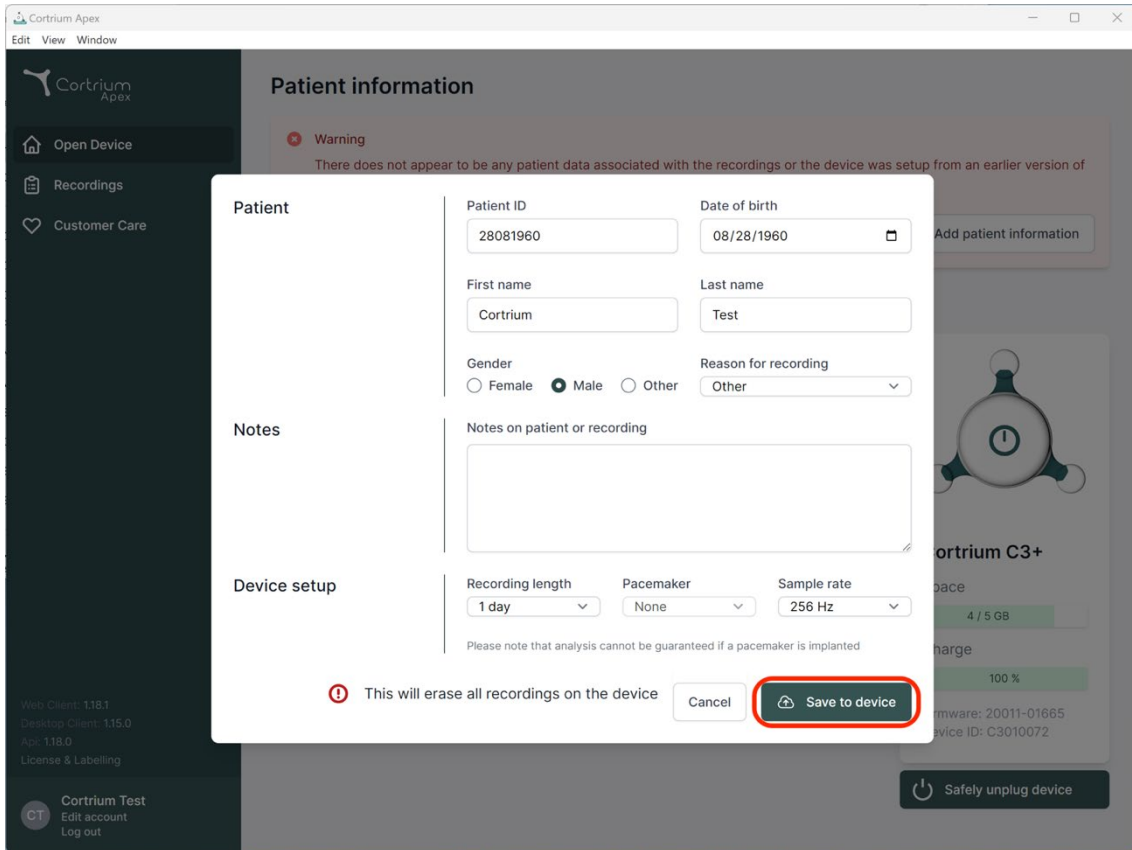Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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This will erase all recordings on the device Cancel **Save to device**

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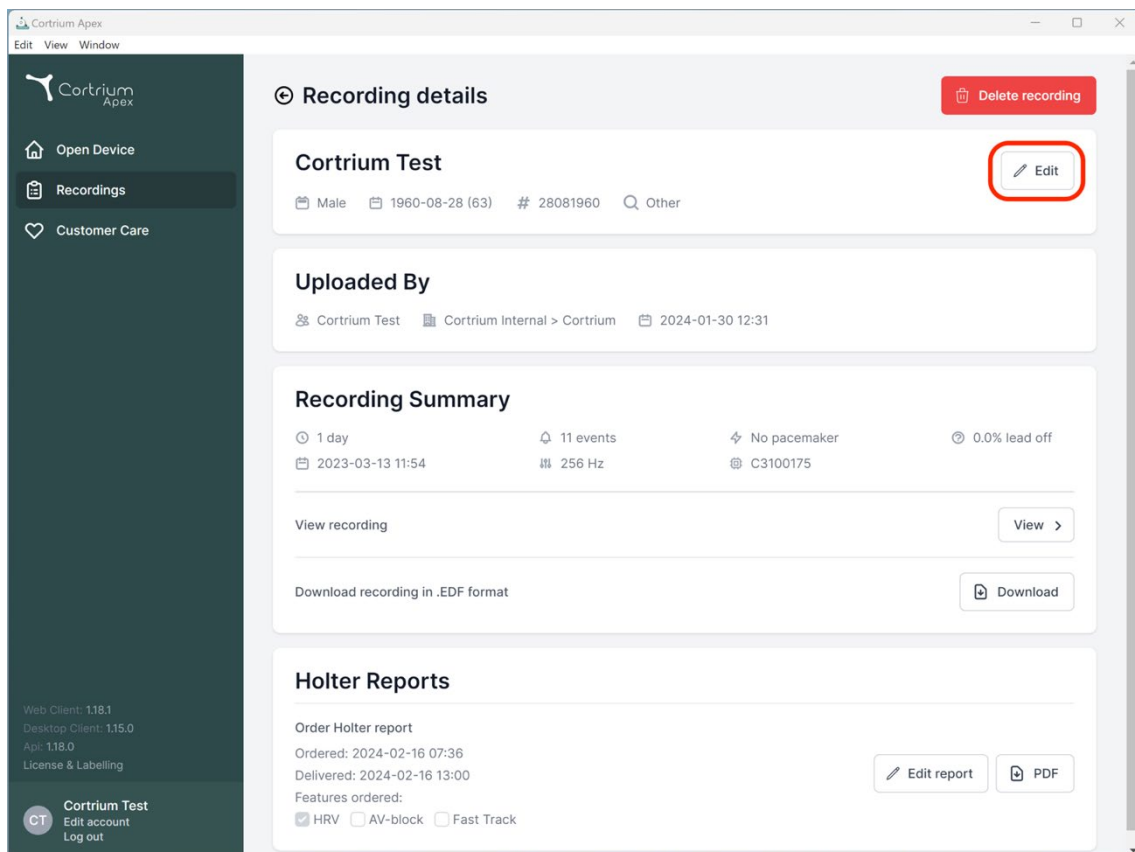
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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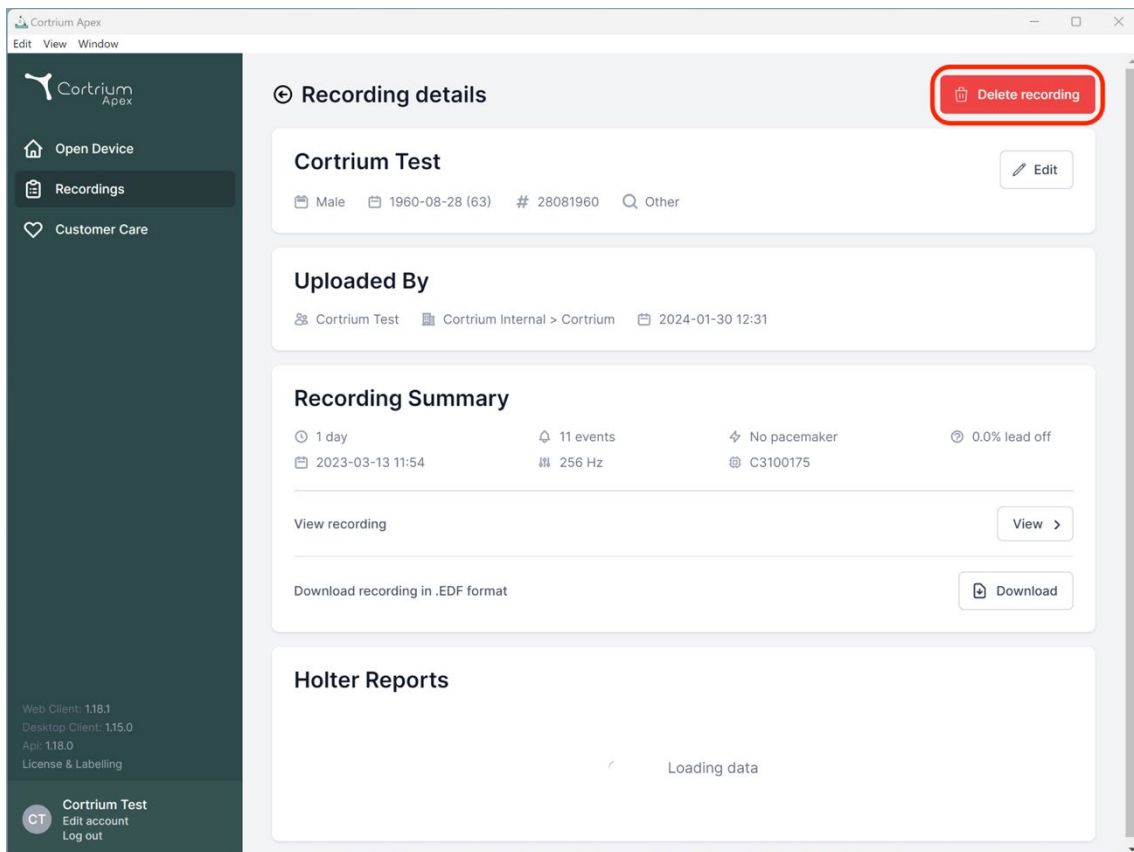
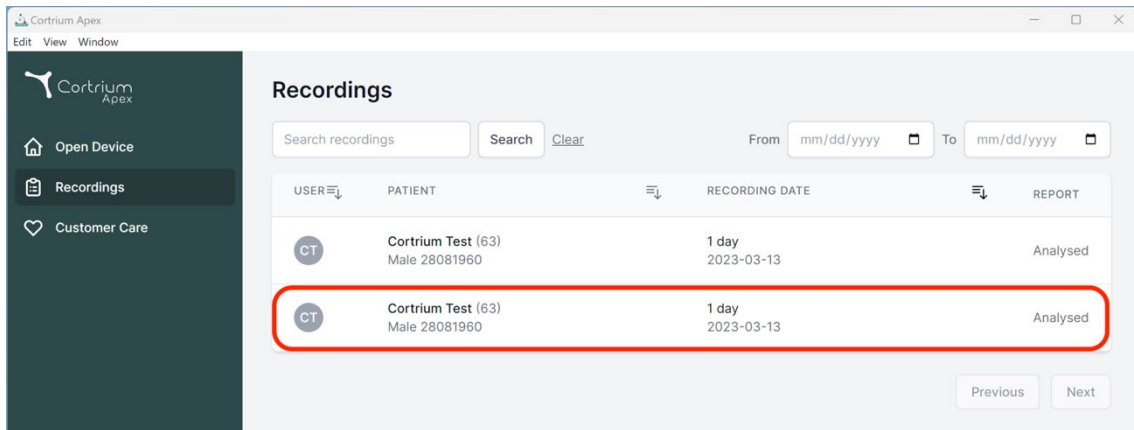
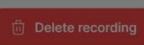
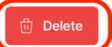
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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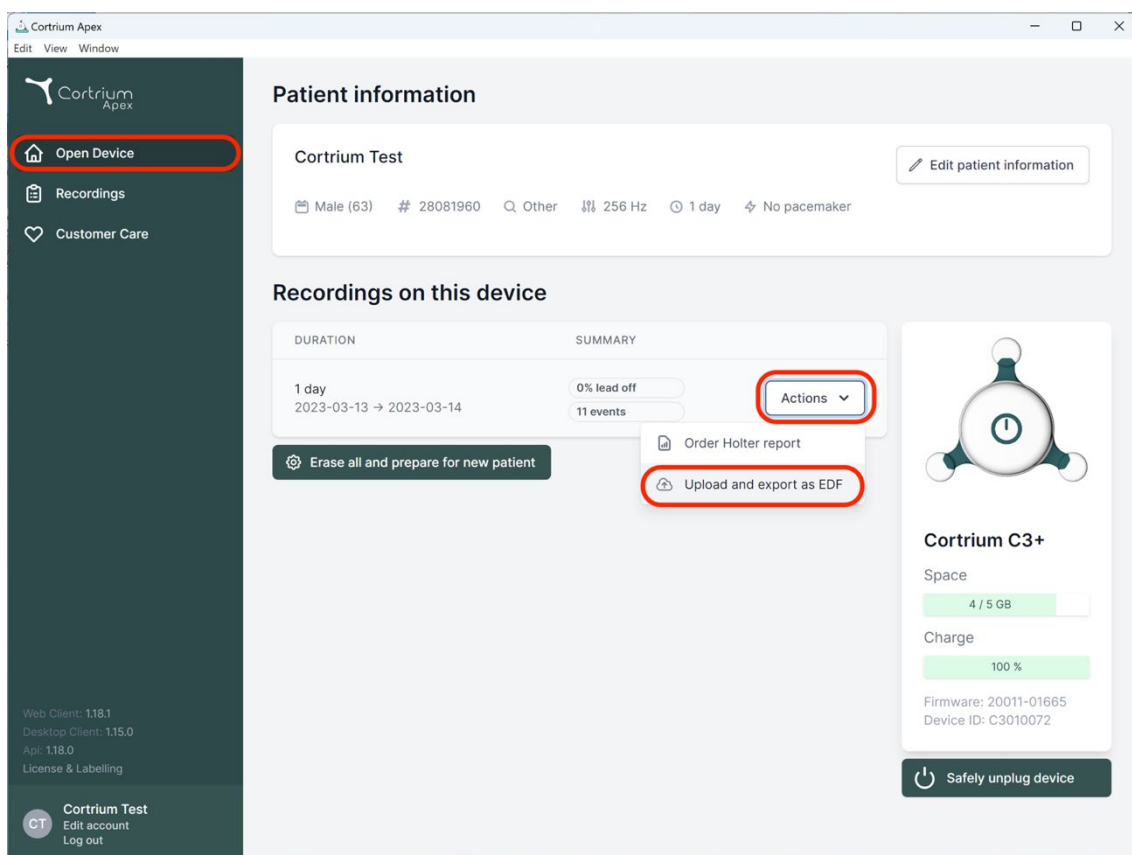
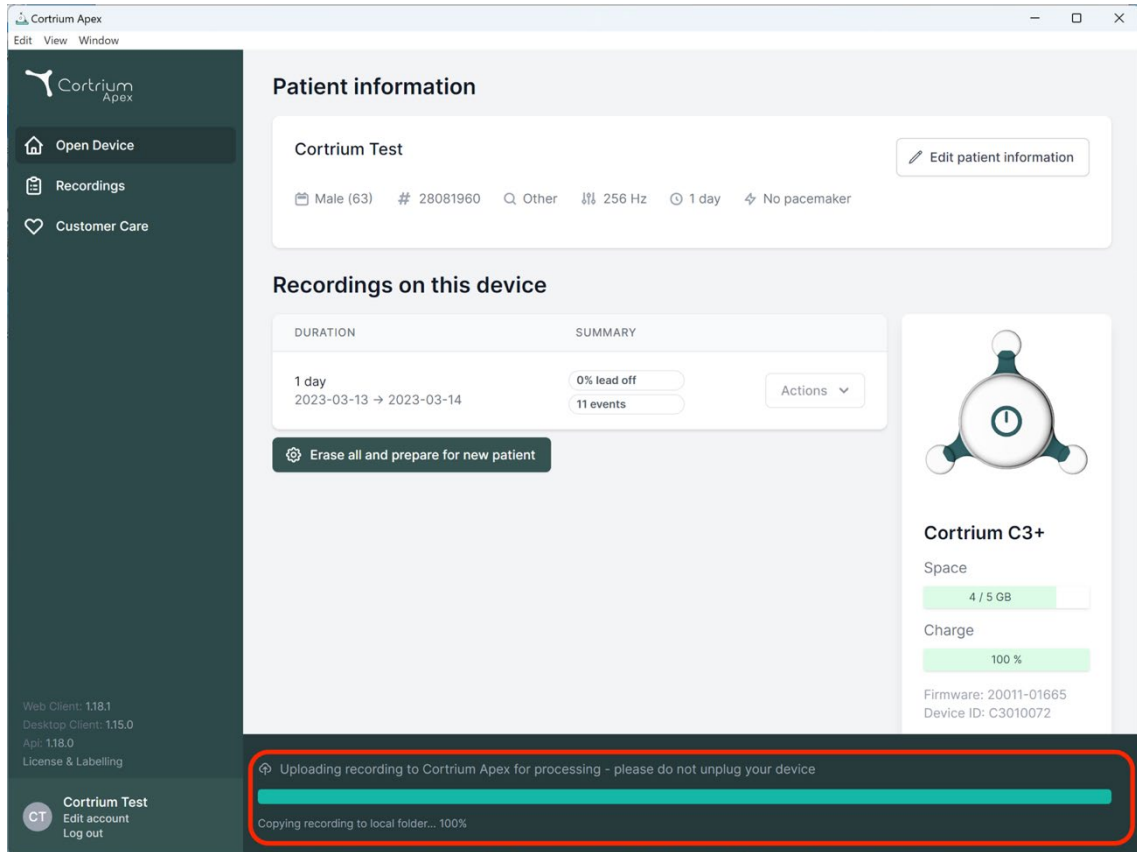
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

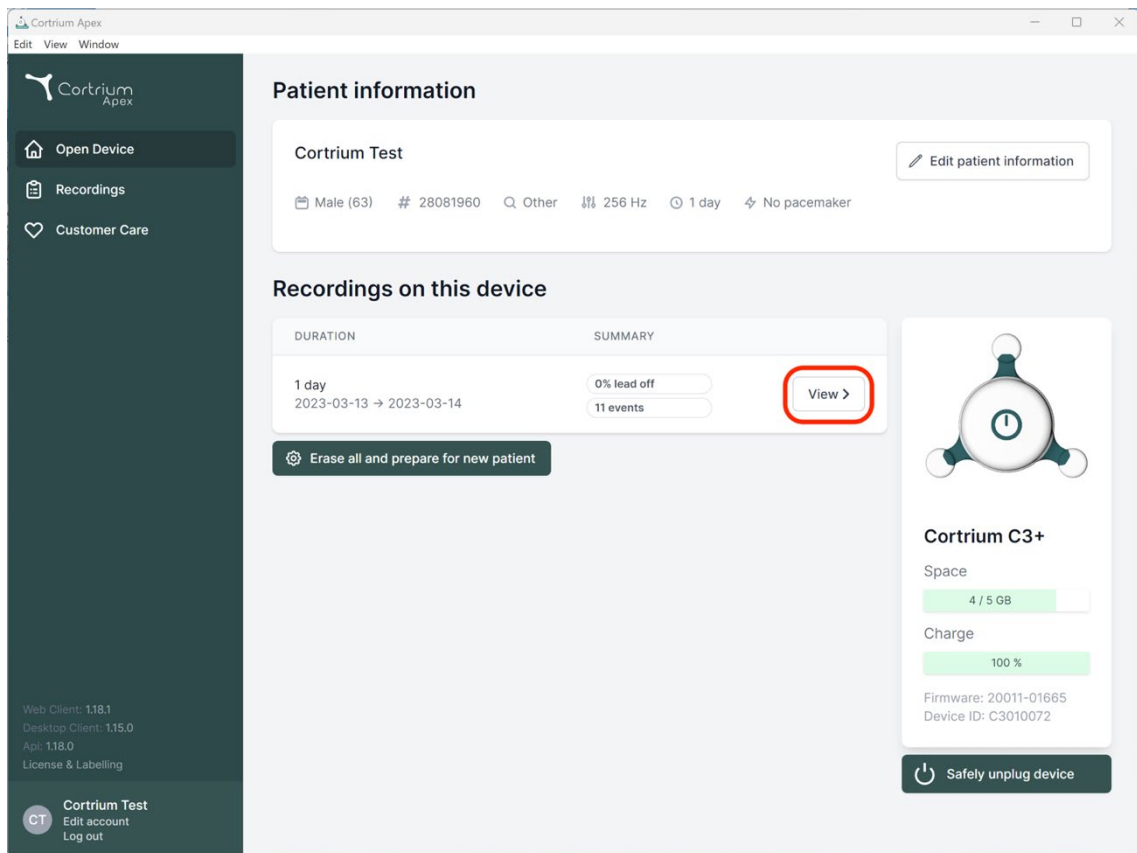
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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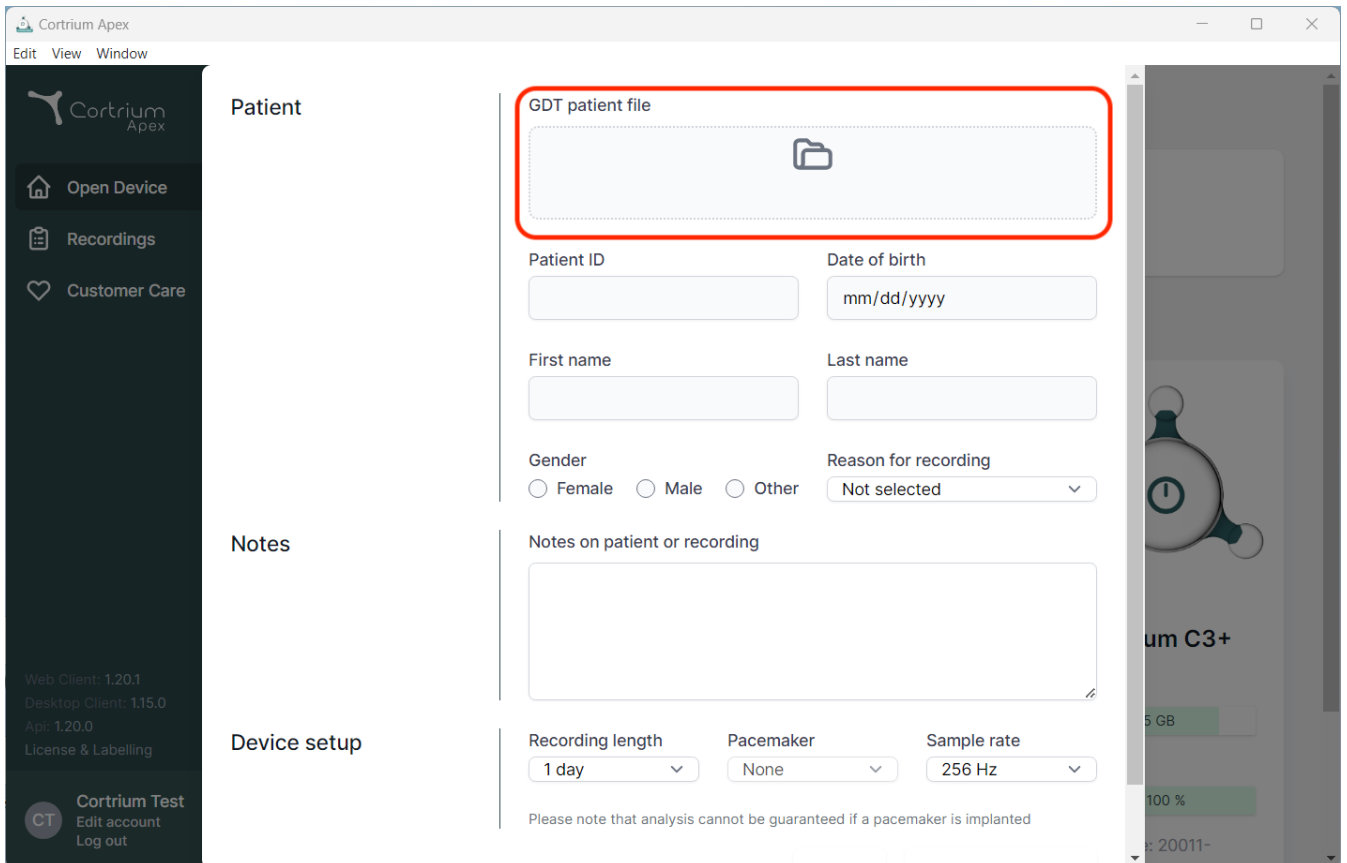
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted and contains a form for entering patient information. The form includes a red-bordered box for the GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. Below these are radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

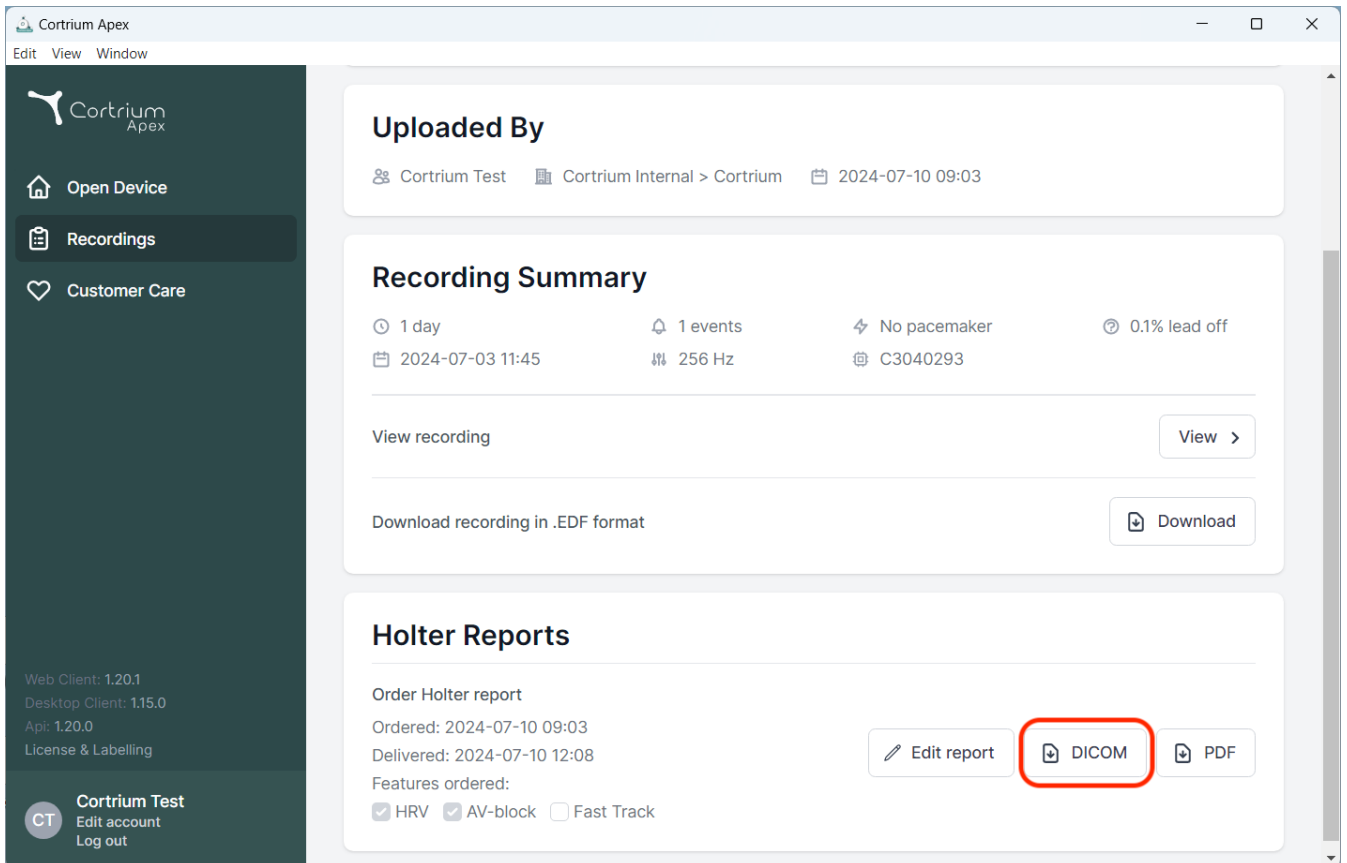
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom left of the sidebar shows version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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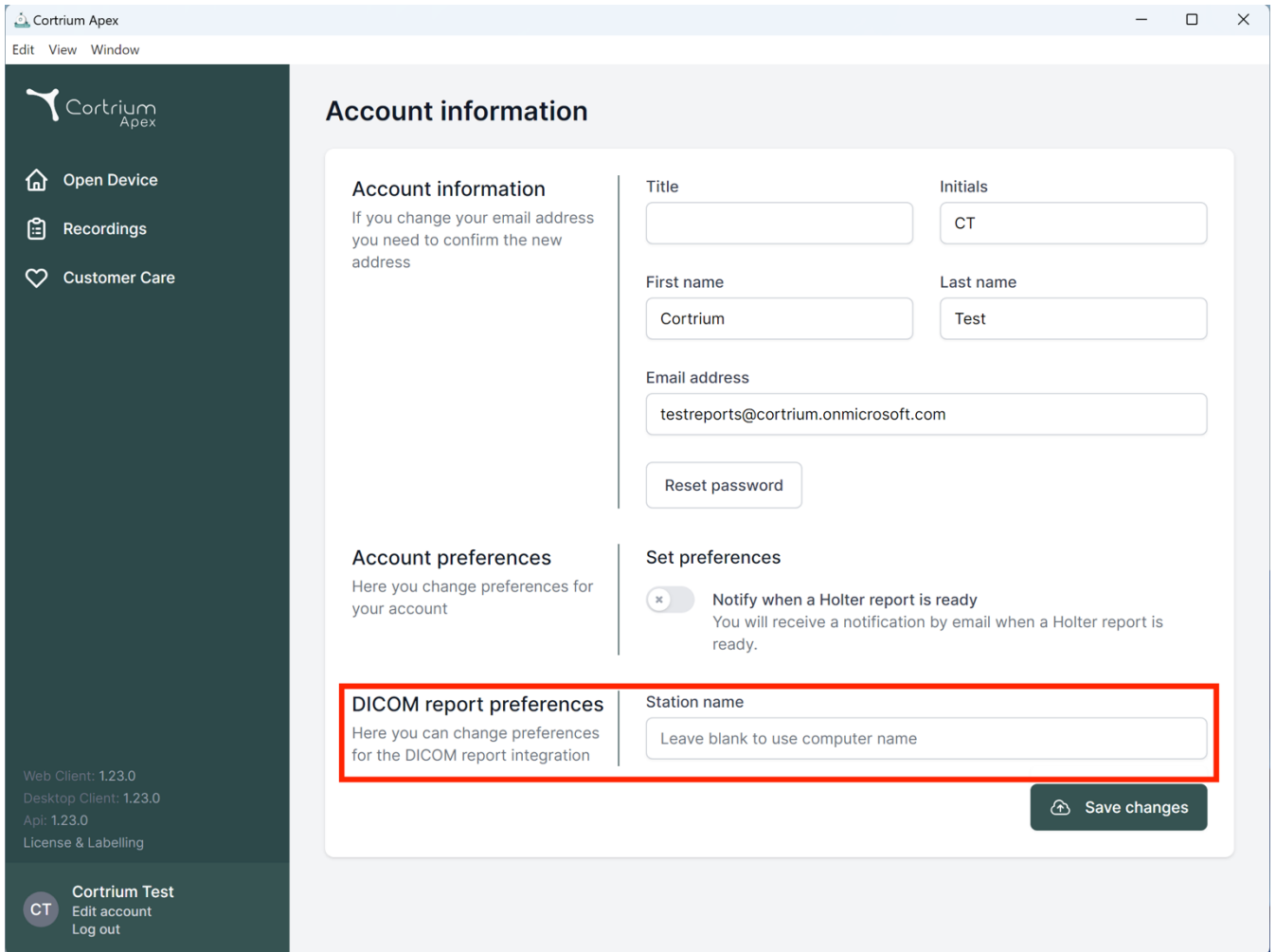Cortrium Apex
Edit View Window

Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

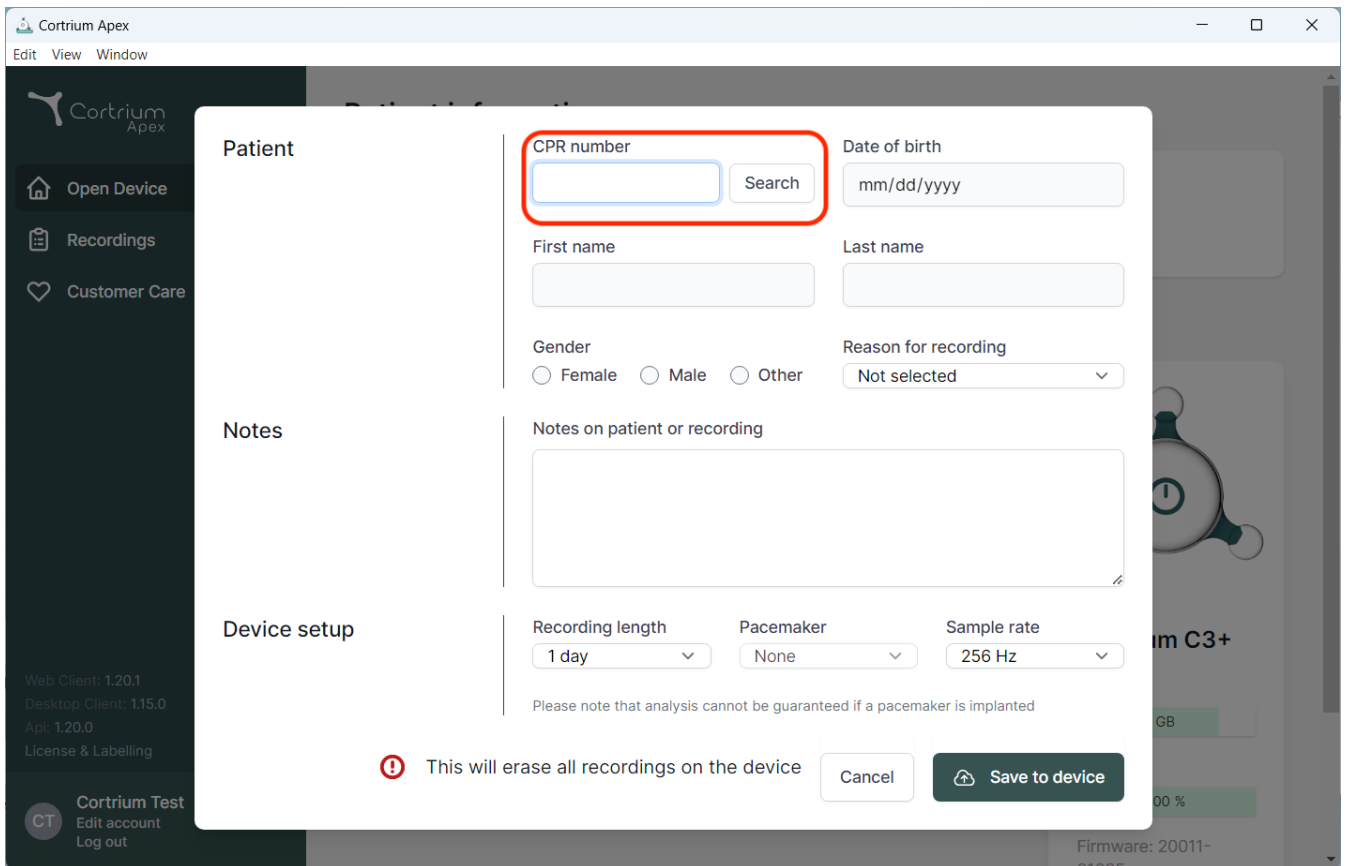
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient search and data entry. The form is divided into several sections:

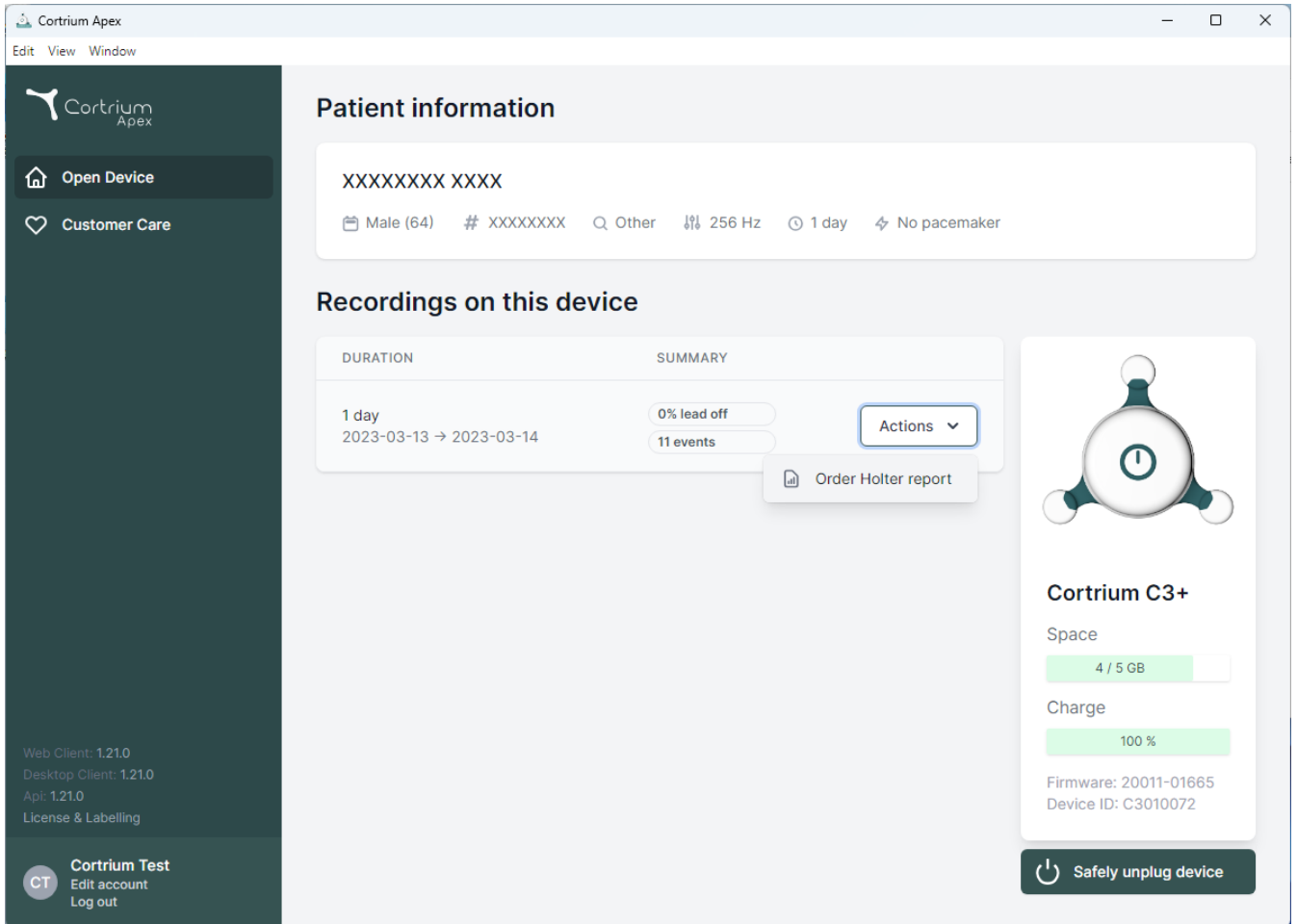
- Patient:** Contains fields for "CPR number" (highlighted with a red box), "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (radio buttons for Female, Male, Other), and "Reason for recording" (a dropdown menu currently showing "Not selected").
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Includes "Recording length" (set to 1 day), "Pacemaker" (set to None), and "Sample rate" (set to 256 Hz).

Below the form, a warning message with a red exclamation mark icon states: "This will erase all recordings on the device". At the bottom right of the modal are two buttons: "Cancel" and "Save to device".

The background shows the Cortrium Apex application menu with options like "Open Device", "Recordings", and "Customer Care". The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test" and "Edit account" / "Log out" options.

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The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', 'Customer Care', and version information (Web Client: 1.21.0, Desktop Client: 1.21.0, Api: 1.21.0, License & Labelling). The main content area is divided into sections: 'Patient information' showing a masked patient ID (XXXXXXX XXXX) and details like 'Male (64)', '256 Hz', '1 day', and 'No pacemaker'; 'Recordings on this device' with a table of recordings (1 day, 2023-03-13 → 2023-03-14) and a summary (0% lead off, 11 events); and a device status panel for 'Cortrium C3+' showing 'Space' (4 / 5 GB), 'Charge' (100 %), and 'Firmware: 20011-01665 Device ID: C3010072'. A 'Safely unplug device' button is at the bottom right.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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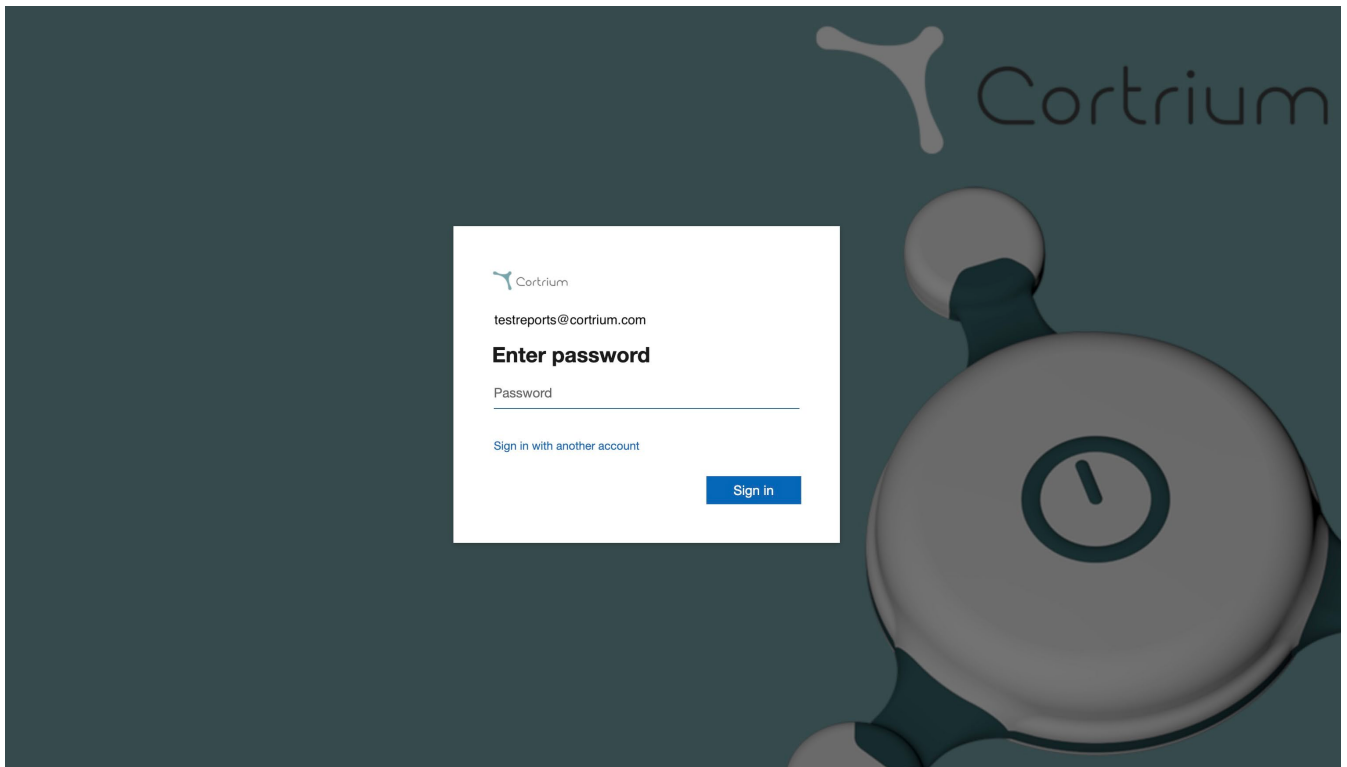
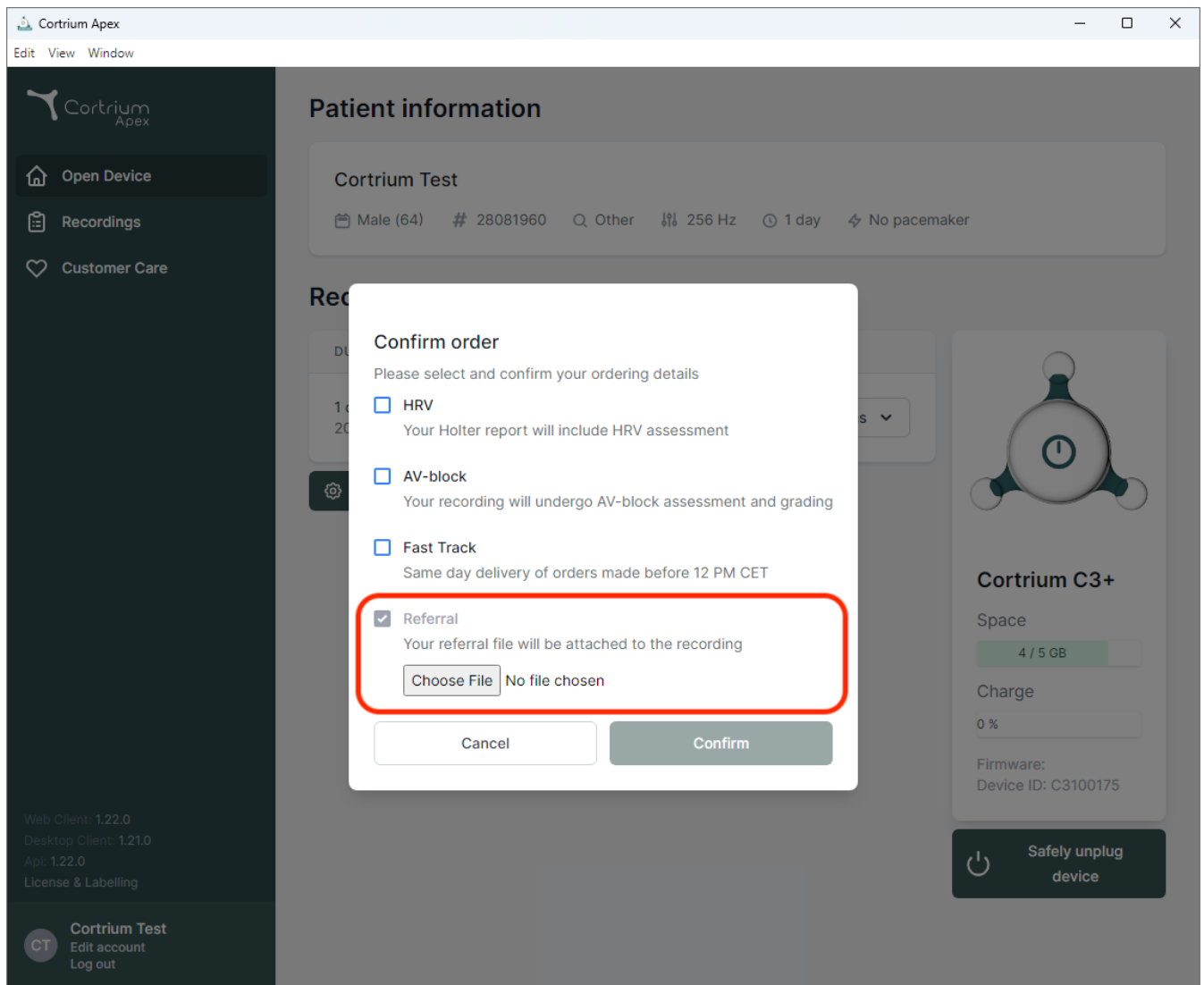
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
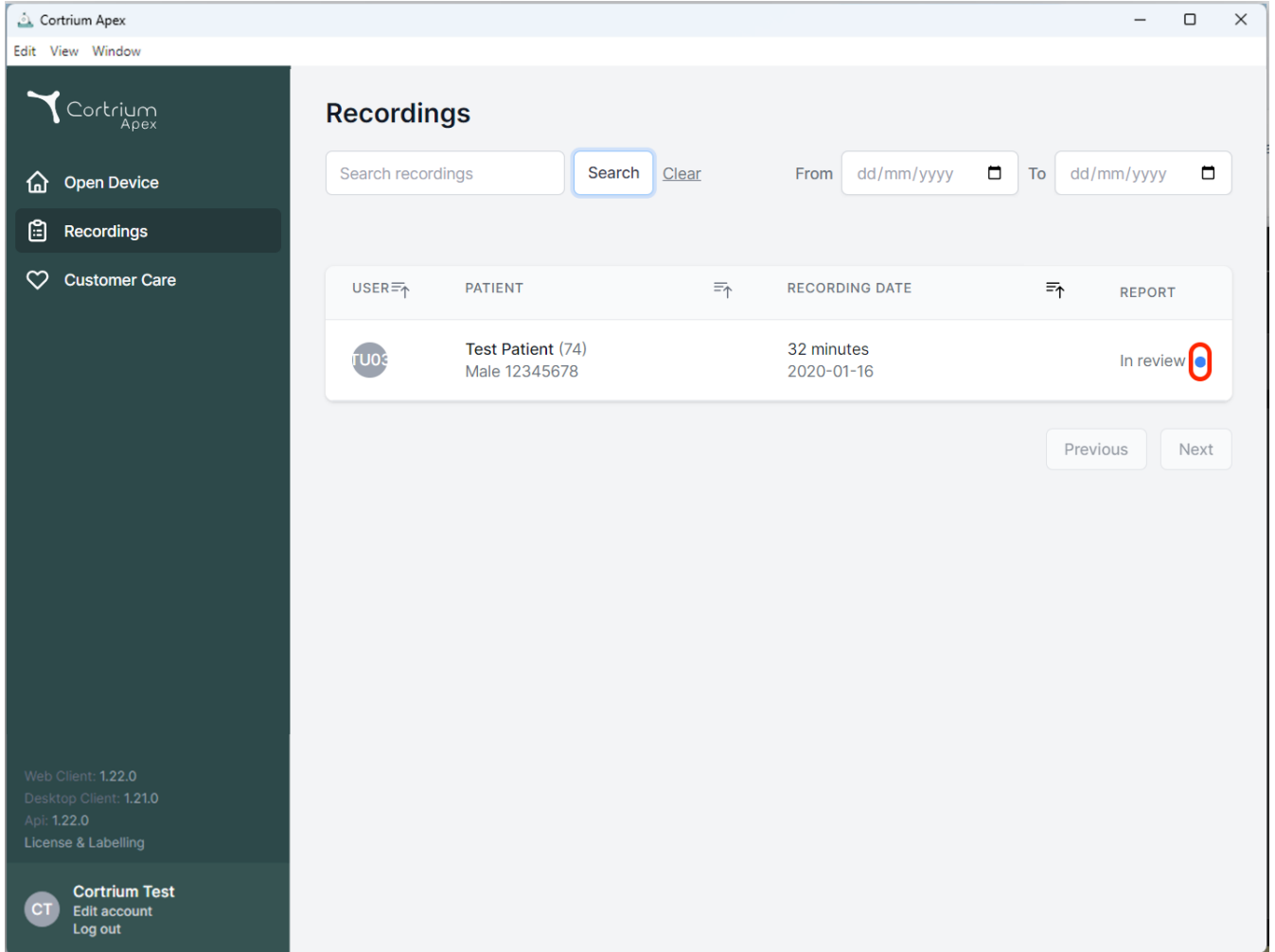
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot next to "In review" is circled in red. At the bottom of the table are "Previous" and "Next" buttons. The footer of the sidebar displays version information: "Web Client: 1.22.0", "Desktop Client: 1.21.0", "Api: 1.22.0", and "License & Labelling". The user profile "Cortrium Test" is also visible.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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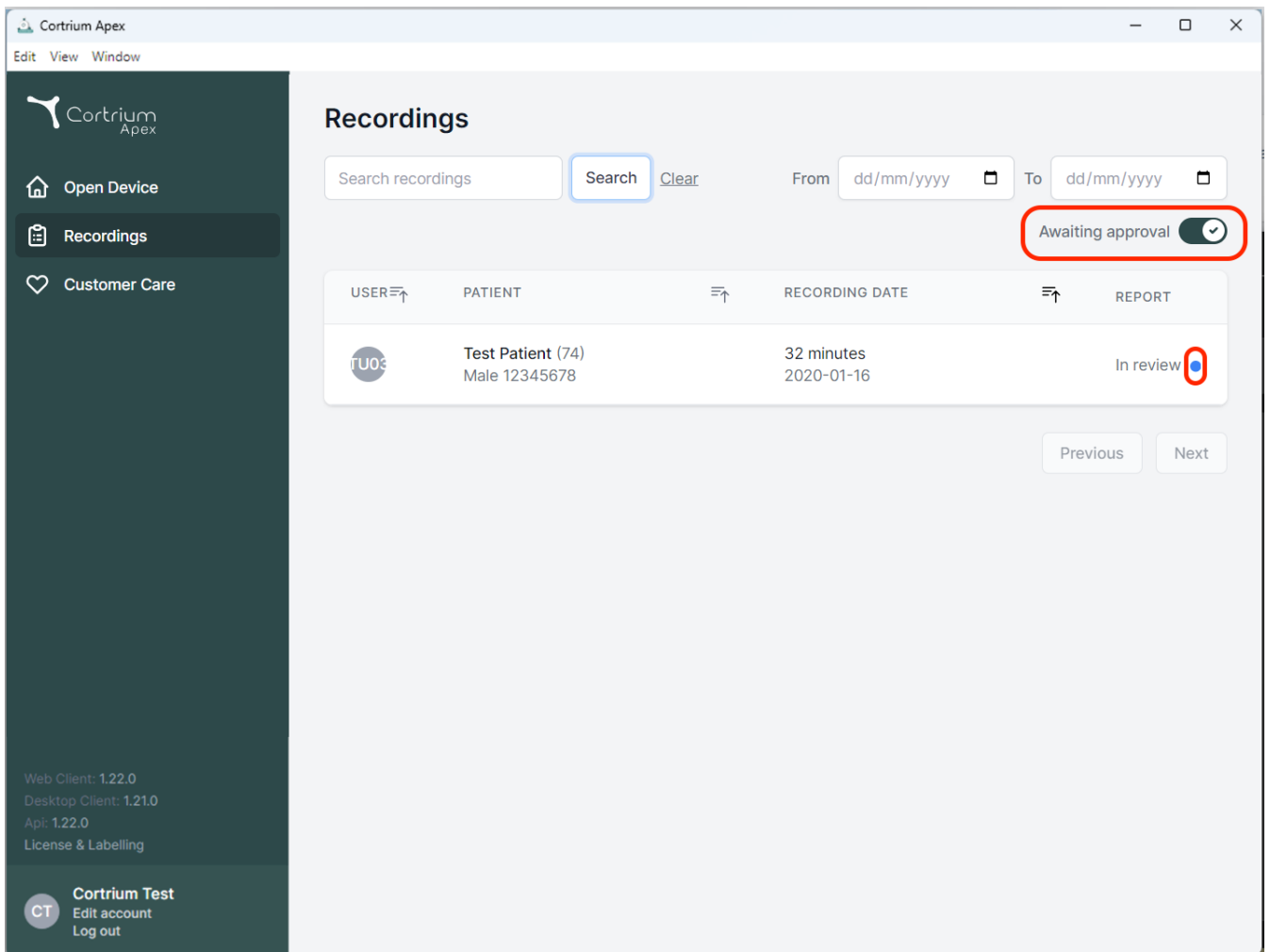
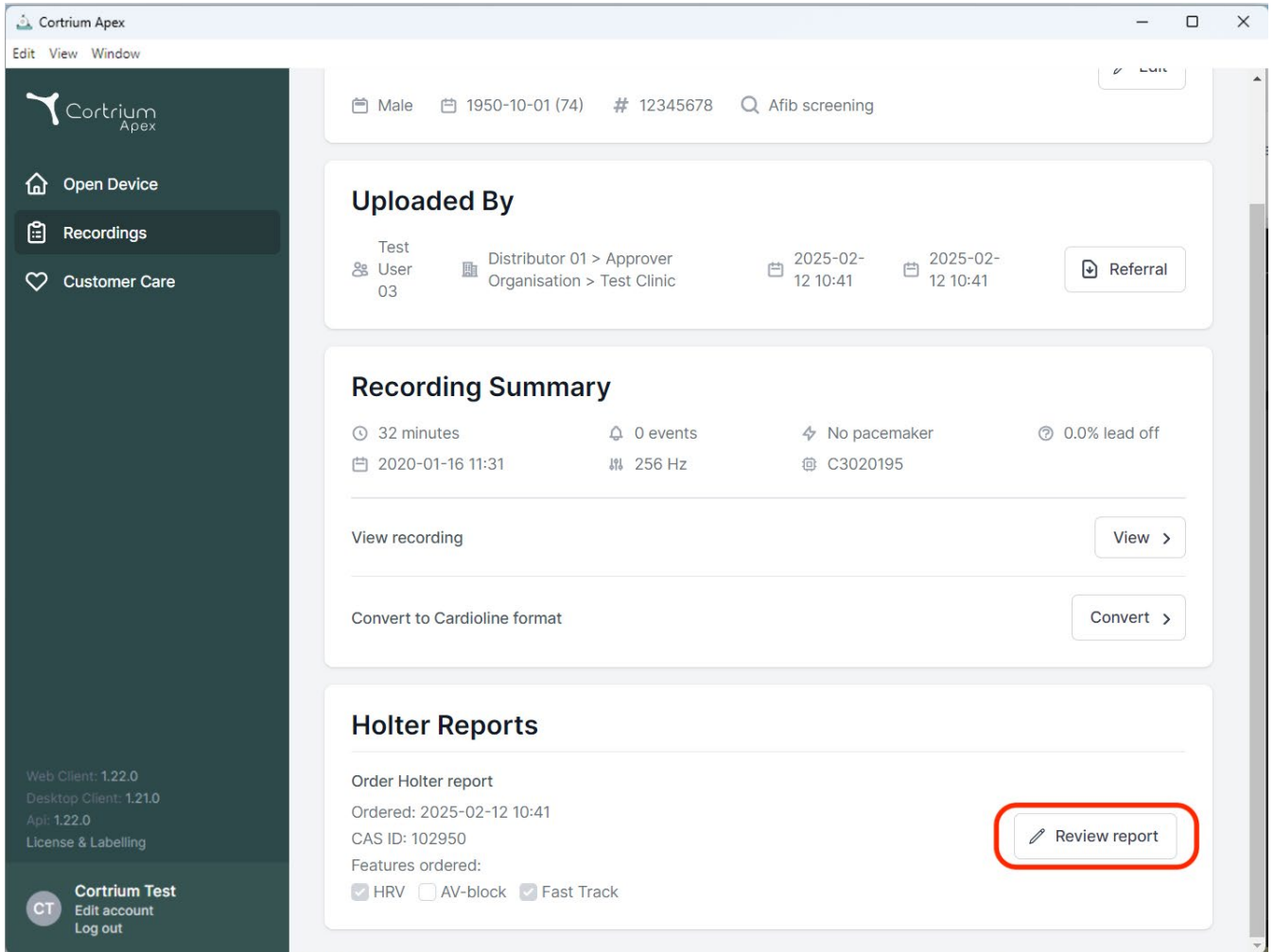
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and three main menu items: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and upload dates (2025-02-12 10:41). A 'Referral' button is present.
- Recording Summary:** A section providing key recording metrics: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date (2020-01-16 11:31) and frequency (256 Hz). Below this, there are buttons for 'View recording' and 'Convert to Cardioline format'.
- Holter Reports:** A section for managing holter reports. It shows the 'Order Holter report' status, the order date (2025-02-12 10:41), and the CAS ID (102950). Under 'Features ordered', there are checkboxes for 'HRV' (checked), 'AV-block' (unchecked), and 'Fast Track' (checked). A red box highlights the 'Review report' button.

At the bottom left, the version information is displayed: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Apl: 1.22.0', and 'License & Labelling'. The bottom right corner shows the user 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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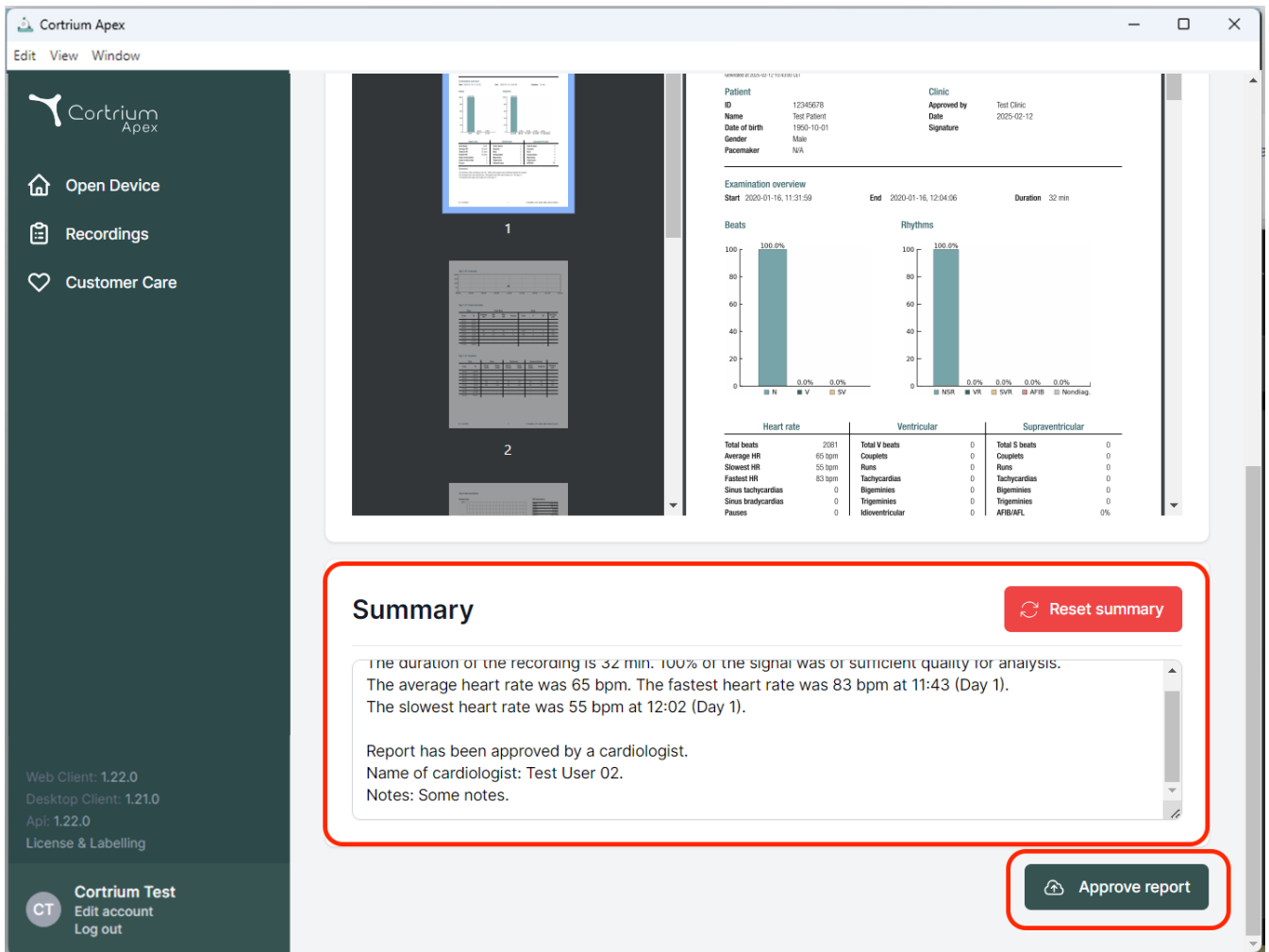Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
App: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

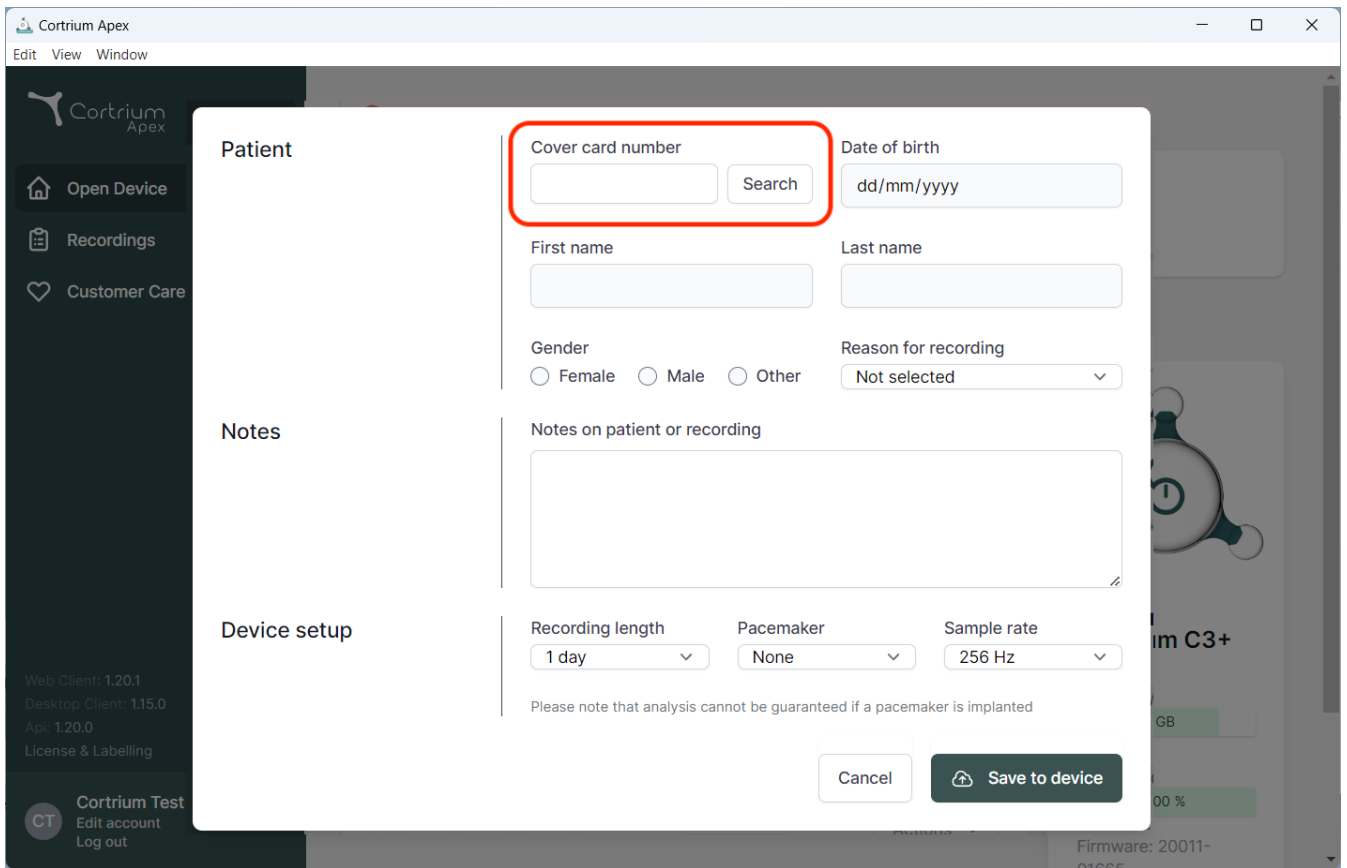
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for patient information. The form includes fields for "Cover card number" (highlighted with a red box), "Date of birth" (format dd/mm/yyyy), "First name", "Last name", "Gender" (radio buttons for Female, Male, Other), "Reason for recording" (dropdown menu), and "Notes on patient or recording" (text area). Below the form, there are settings for "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A disclaimer states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right, there are "Cancel" and "Save to device" buttons. The background shows the main interface with a sidebar containing "Open Device", "Recordings", and "Customer Care" options, and a footer with version information and a "Cortrium Test" button.

Patient

Cover card number Search

Date of birth

First name

Last name

Gender
☐ Female ☐ Male ☐ Other

Reason for recording
Not selected

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel Save to device