# Cortrium Apex 1.26

# Cortrium Apex 1.26

## Mode d'emploi

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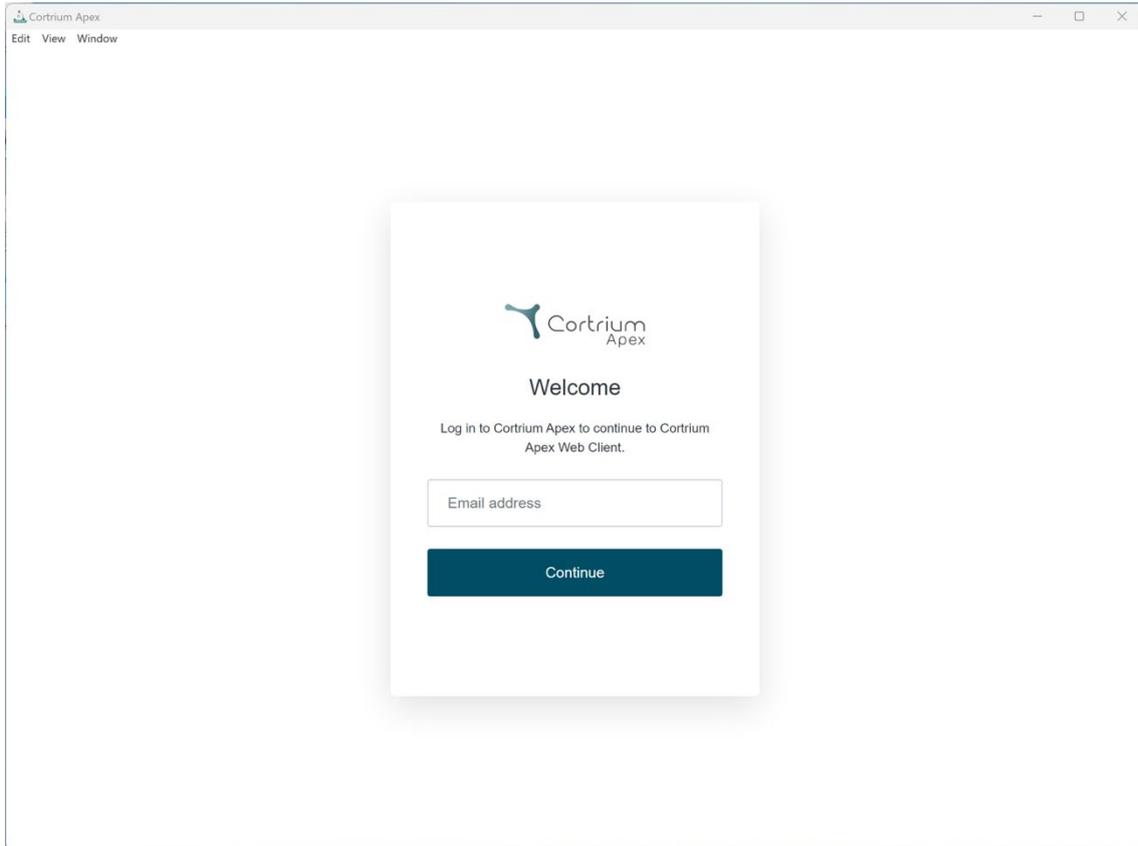
 Cortrium  
Apex

Welcome


Log in to Cortrium Apex to continue to Cortrium  
Apex Web Client.

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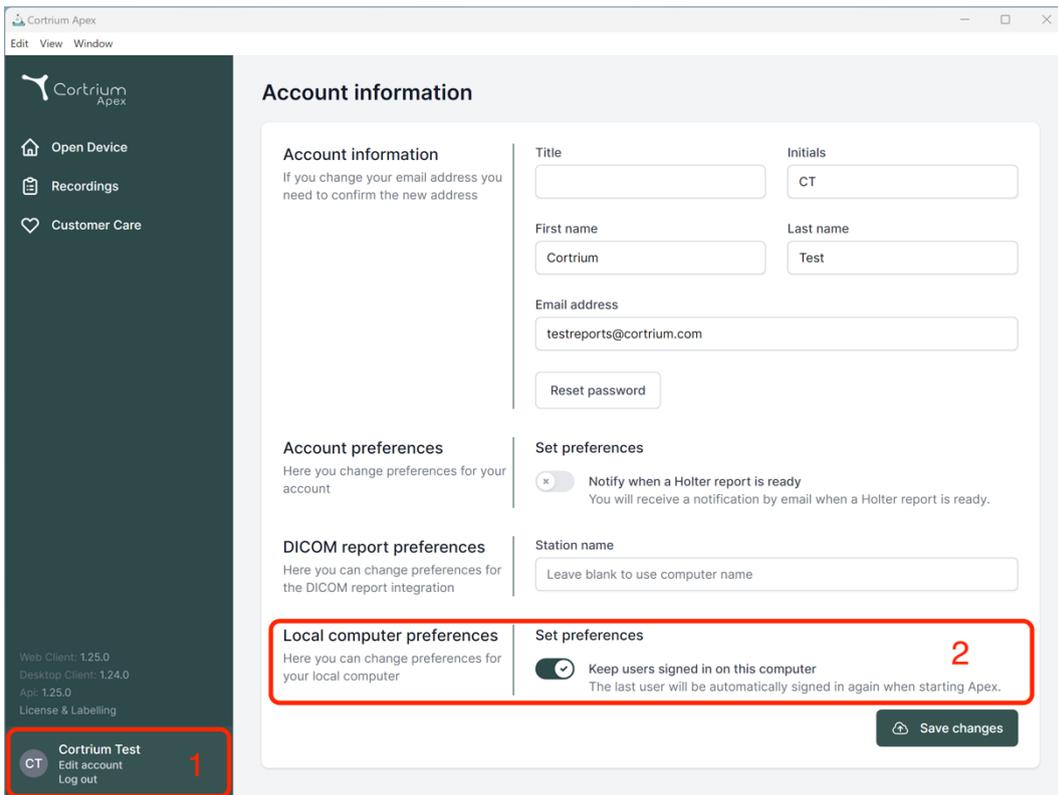#### Local computer preferences

Here you can change preferences for your local computer

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Station name:

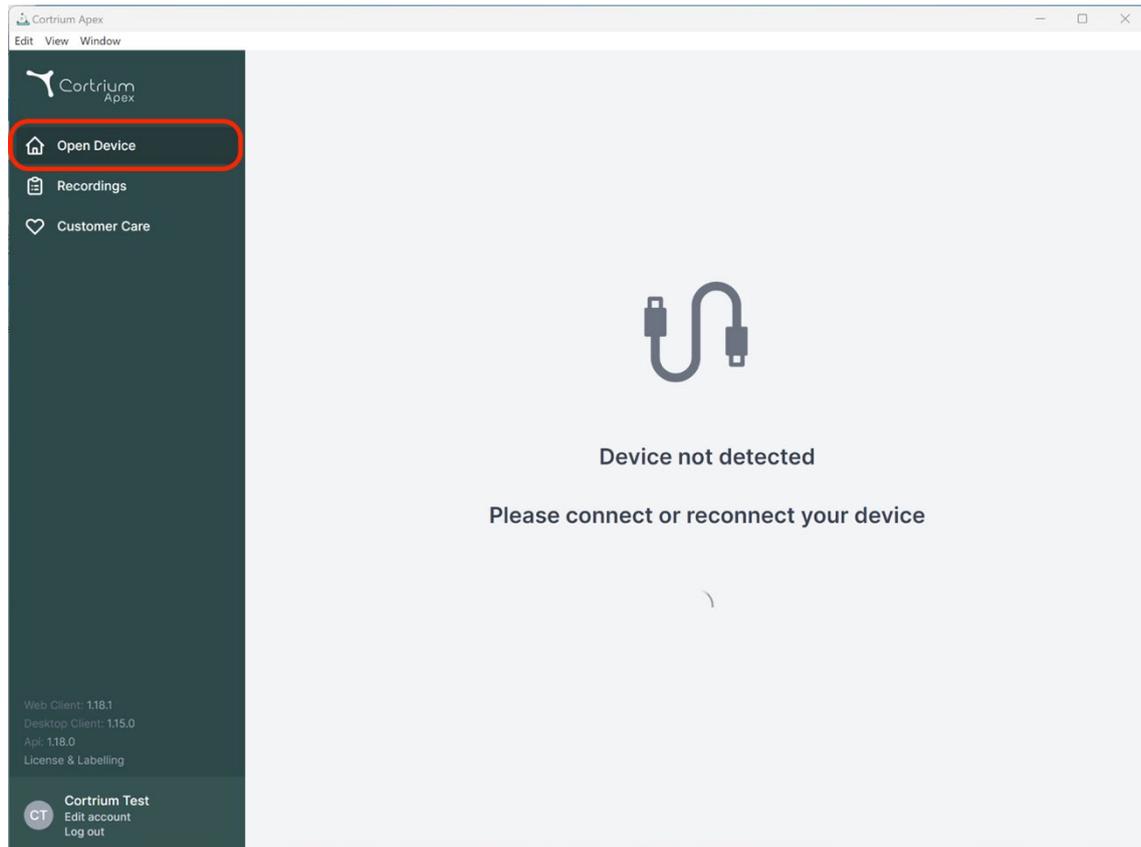
#### Set preferences

Keep users signed in on this computer  
The last user will be automatically signed in again when starting Apex.

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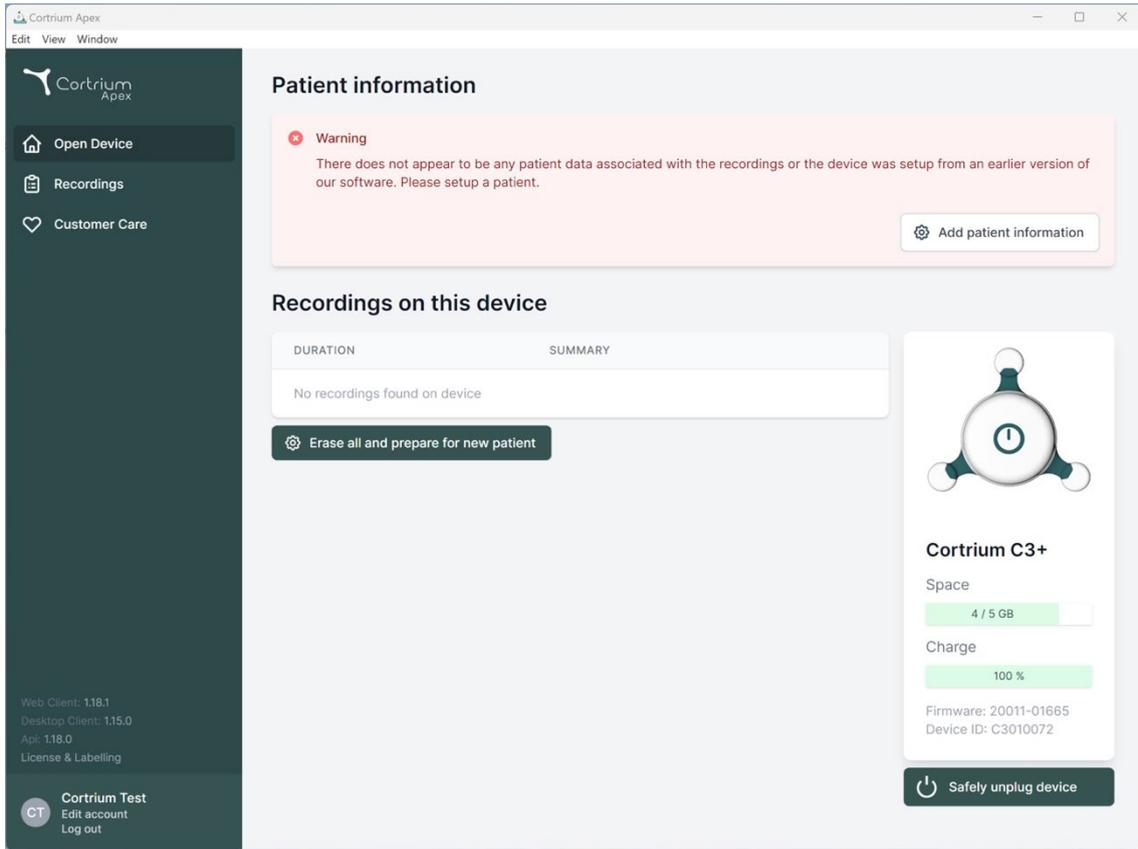
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client (1.18.1), Desktop Client (1.15.0), and App (1.18.0), along with 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table header for 'DURATION' and 'SUMMARY', containing the text 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB, 'Charge' at 100%, and 'Firmware: 20011-01665 Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings
- Customer Care

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**

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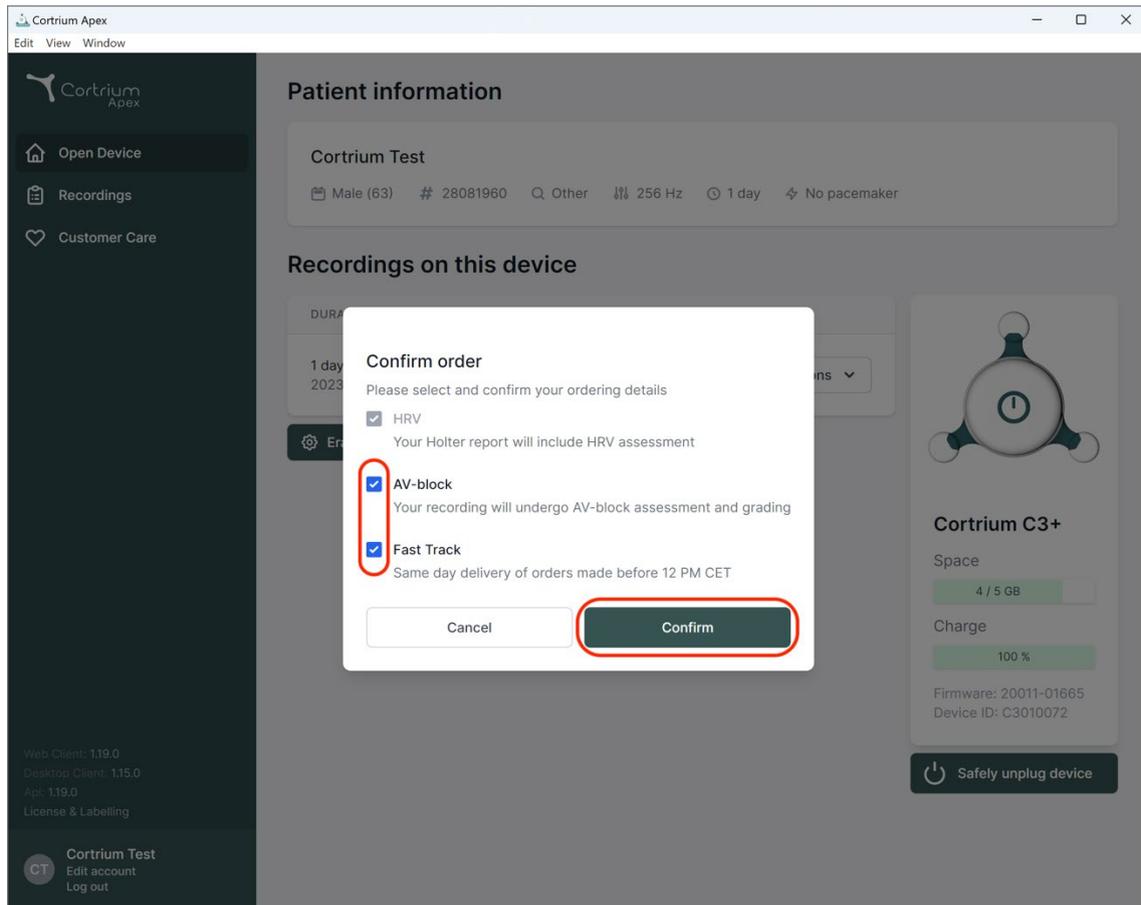
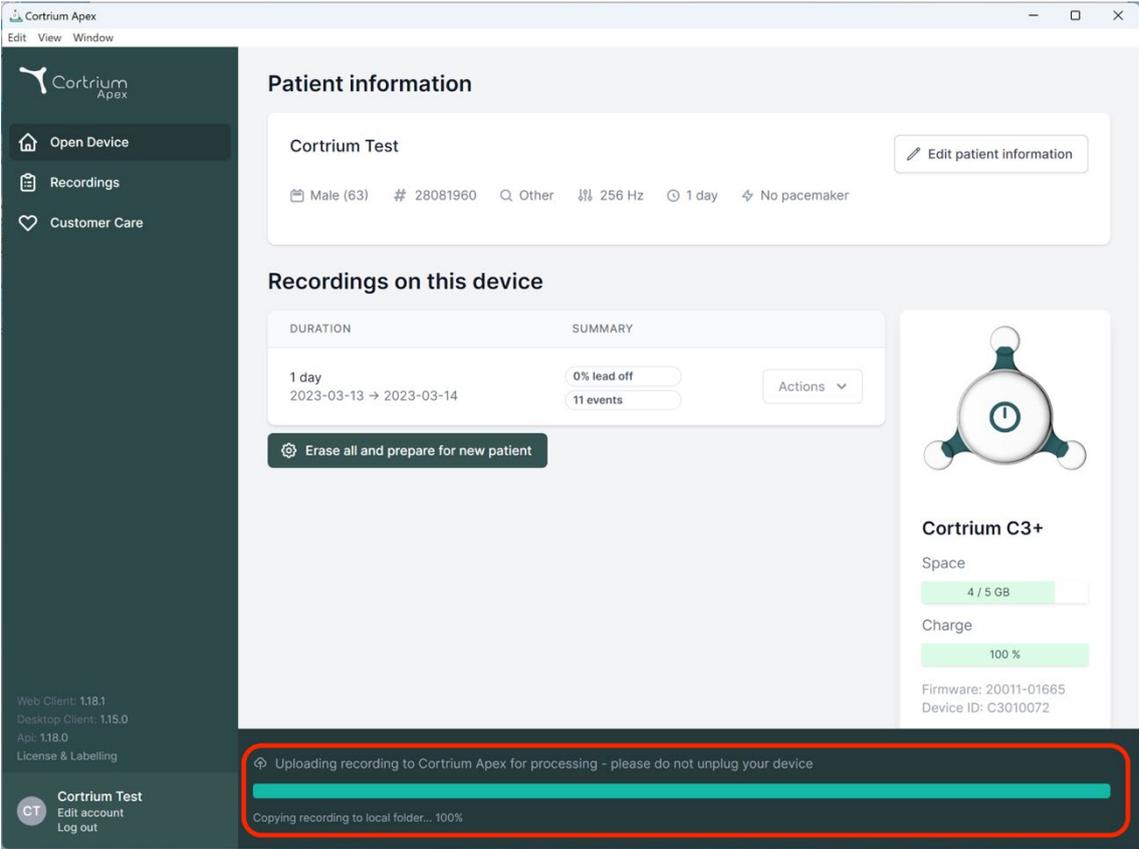
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, a 'Recordings on this device' section shows a table with columns for 'DURATION' and 'SUMMARY'. The recording summary indicates '0% lead off' and '11 events'. A 'Copy' icon and a red-bordered box highlight the status 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar for 'Copying recording to local folder... 100%'. On the right, a 'Cortrium C3+' device status card shows 'Space: 4 / 5 GB' and 'Charge: 100%'. The bottom left of the sidebar contains version information and user options like 'Edit account' and 'Log out'.

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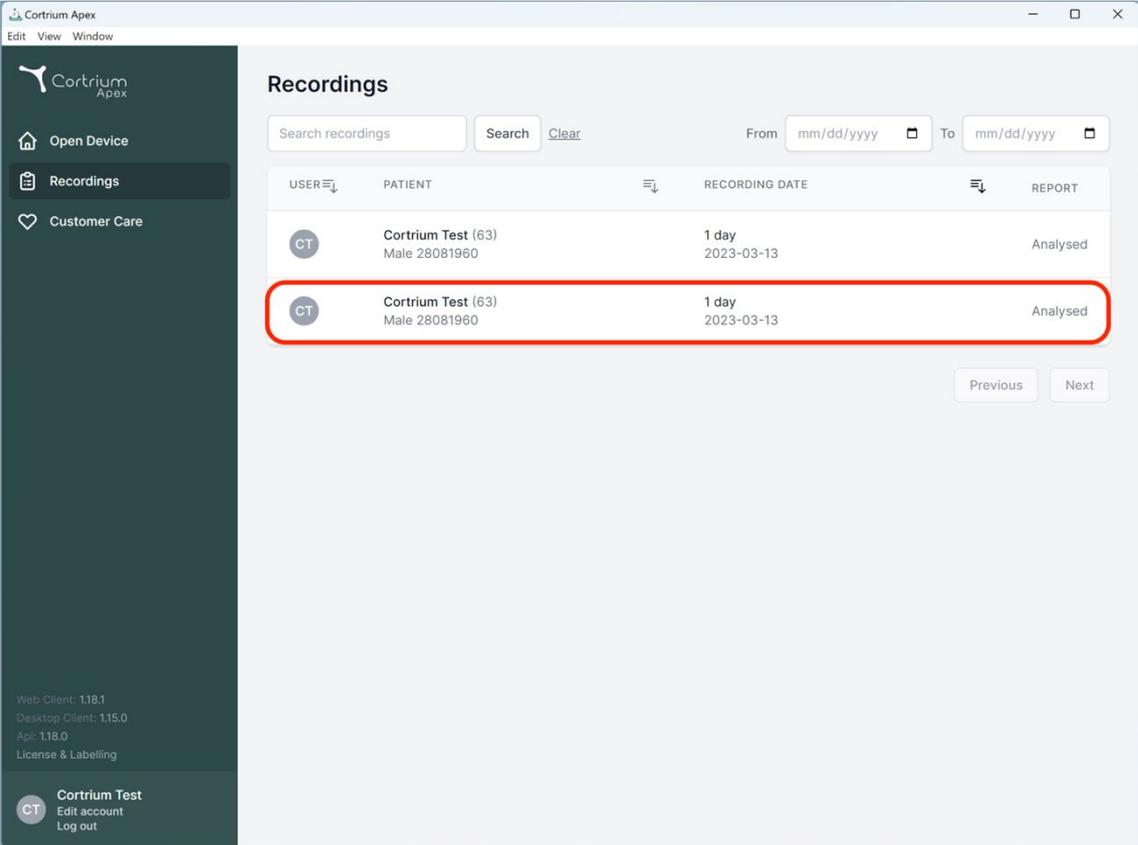
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings (selected), and Customer Care. At the bottom of the sidebar, it shows the user's name 'Cortrium Test' and options to 'Edit account' and 'Log out'. The main content area is titled 'Recordings' and features a search bar, a 'Search' button, a 'Clear' button, and date range filters. Below this is a table with the following data:

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At the bottom right of the table, there are 'Previous' and 'Next' navigation buttons.

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Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track

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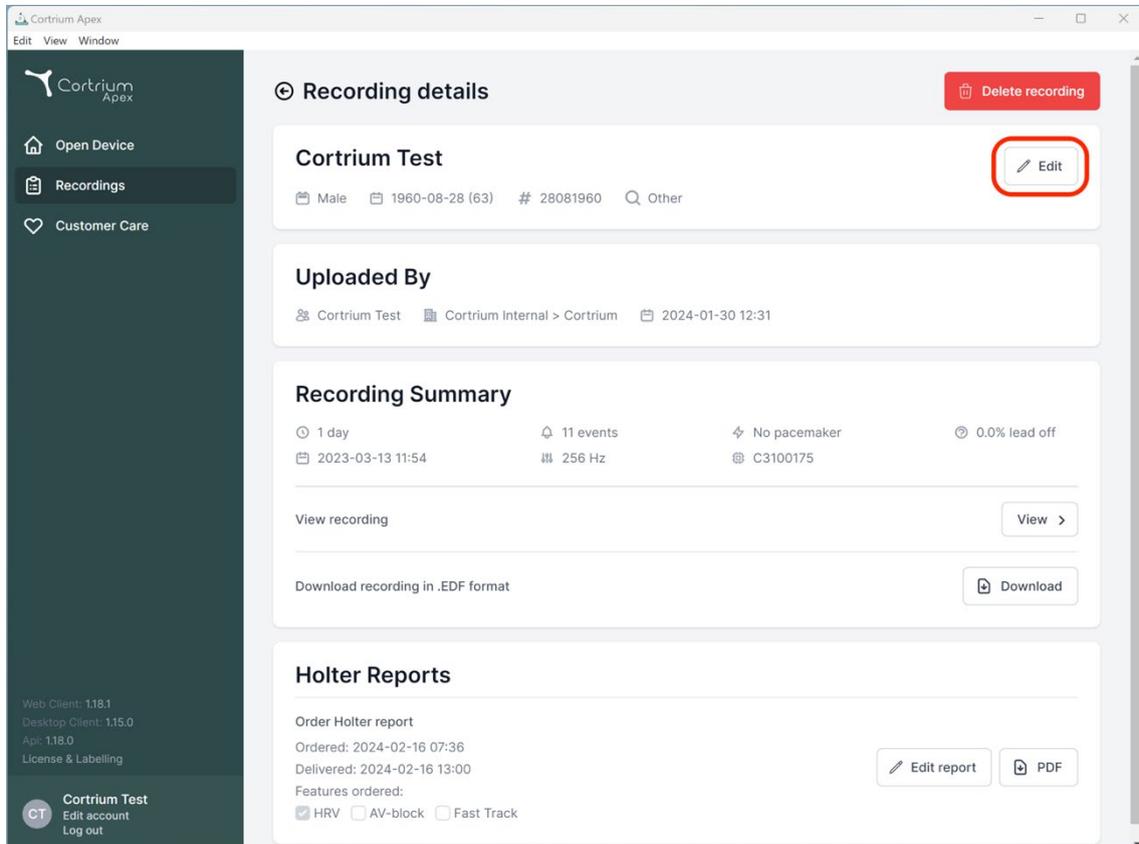
**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**  
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

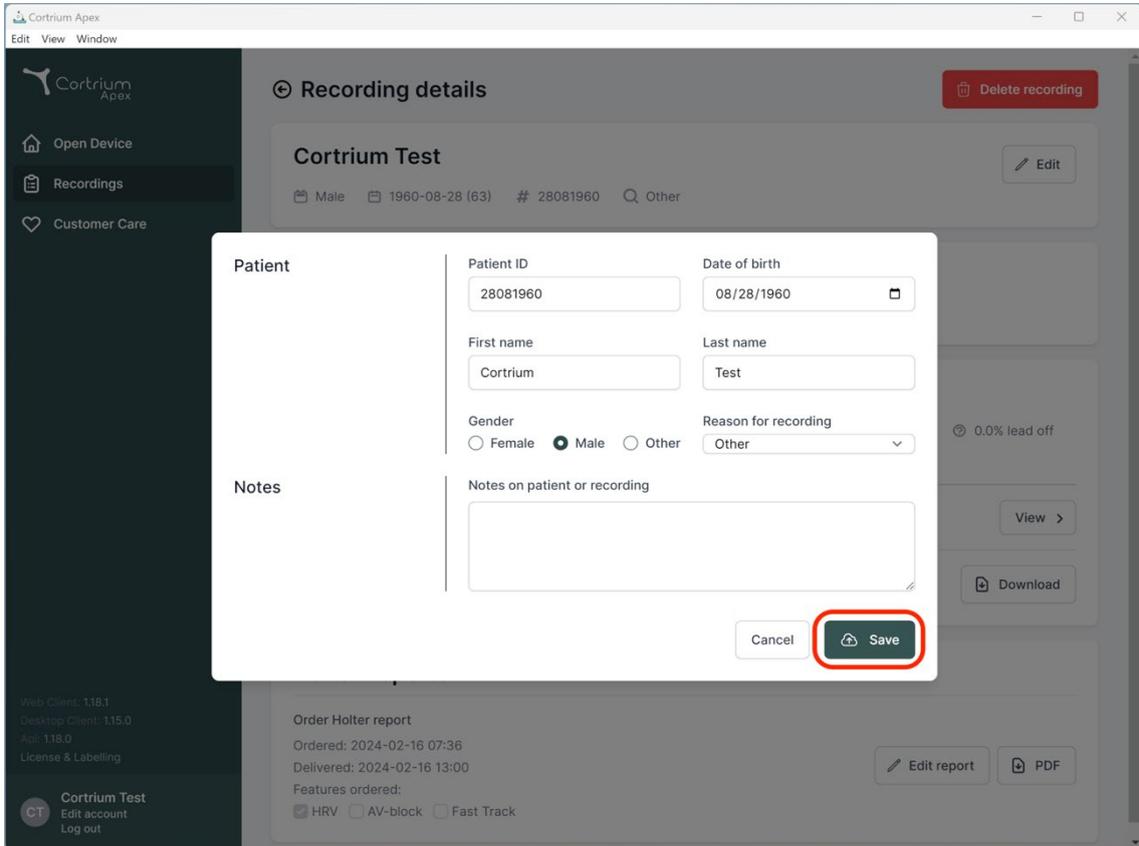
Download recording in .EDF format Download

**Holter Reports**

Order Holter report  
Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track Edit report PDF

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The screenshot displays the 'Recording details' modal in the Cortrium Apex application. The modal is titled 'Cortrium Test' and contains the following fields and options:


- Patient ID:** 28081960
- Date of birth:** 08/28/1960
- First name:** Cortrium
- Last name:** Test
- Gender:**  Female  Male  Other
- Reason for recording:** Other (dropdown menu)
- Notes:** A text area for 'Notes on patient or recording'.

At the bottom of the modal, there are two buttons: 'Cancel' and 'Save'. The 'Save' button is highlighted with a red circle, indicating the action to save the changes.

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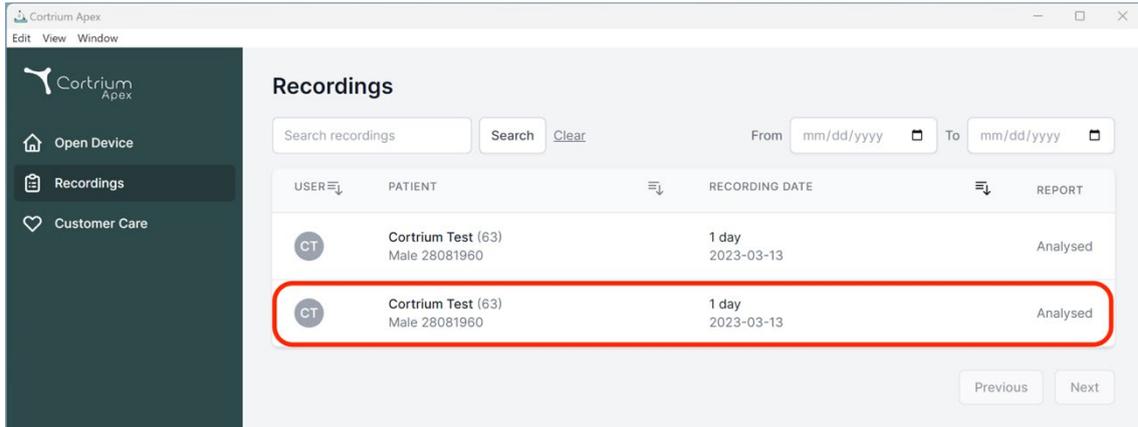
| USER | PATIENT                             | RECORDING DATE      | REPORT   |
|------|-------------------------------------|---------------------|----------|
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |

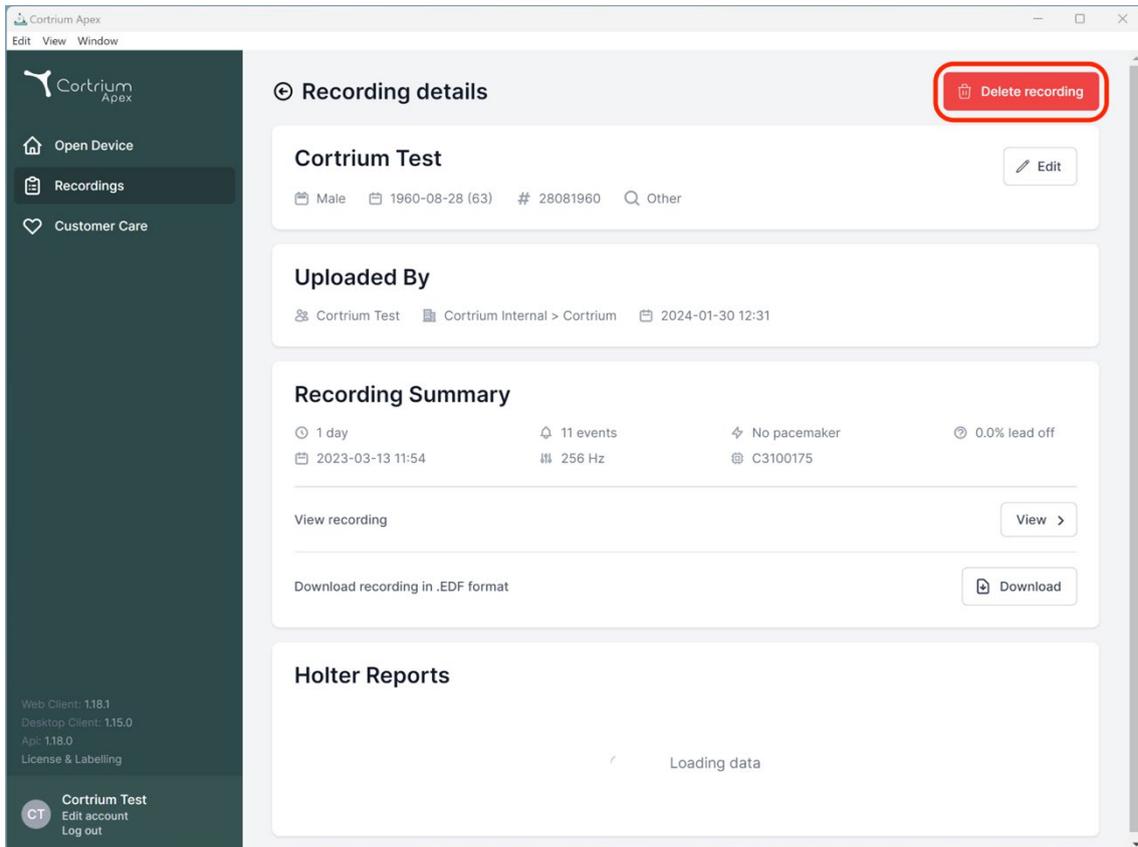**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

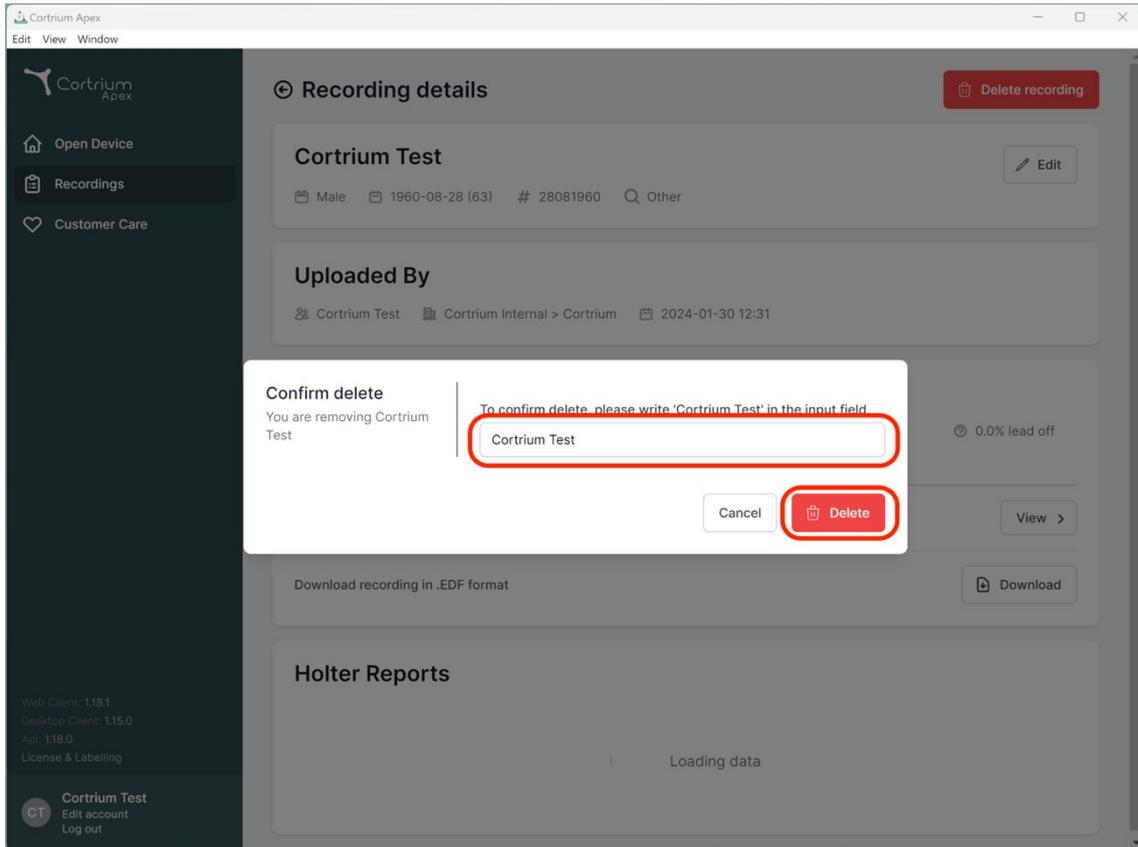
1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

**Holter Reports**

Loading data

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

### Recording details

Delete recording

#### Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Edit

#### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

0.0% lead off

View >

Download recording in .EDF format

Download

#### Holter Reports

Loading data

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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
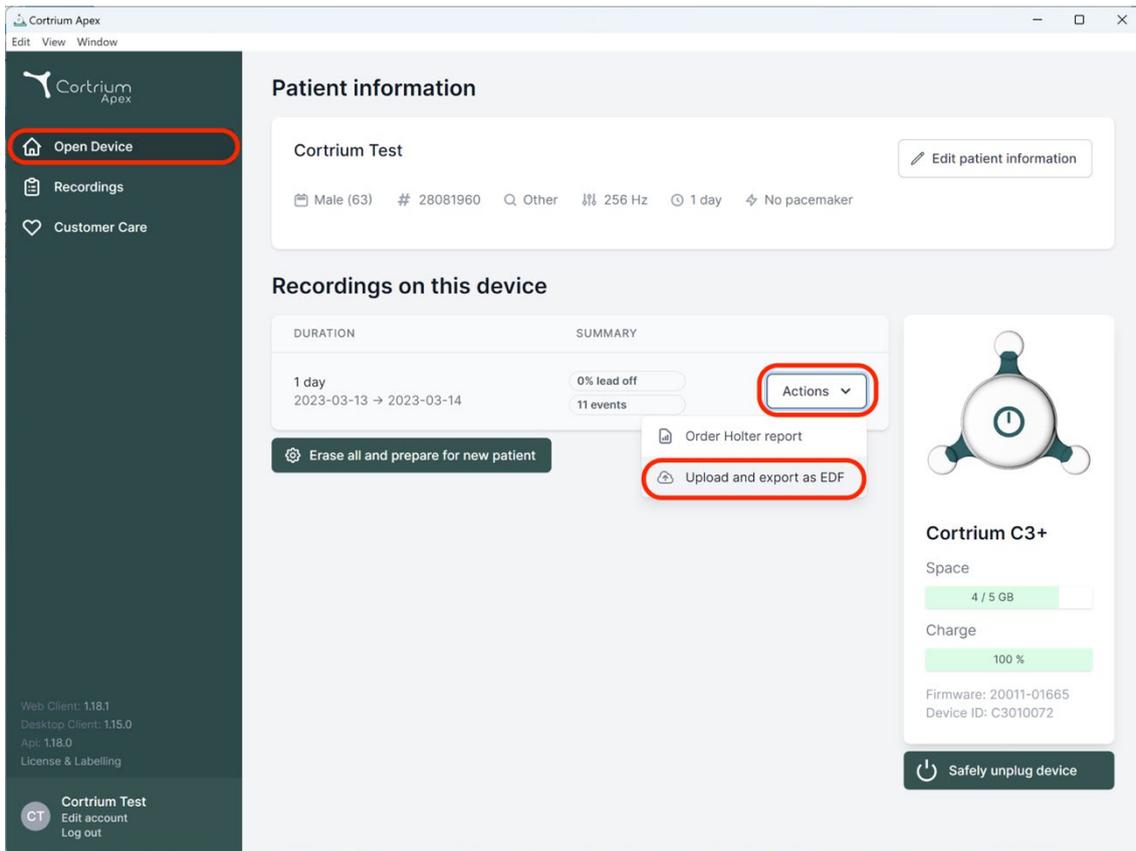
The screenshot shows the Cortrium Apex web application interface. On the left, a dark sidebar contains navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including patient demographics and device settings. Below this, the 'Recordings on this device' section shows a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown menu (highlighted with a red box) is open, showing options like 'Order Holter report' and 'Upload and export as EDF' (highlighted with a red box). On the right, a 'Cortrium C3+' device status card shows 'Space: 4 / 5 GB' and 'Charge: 100%'. A 'Safely unplug device' button is at the bottom right.

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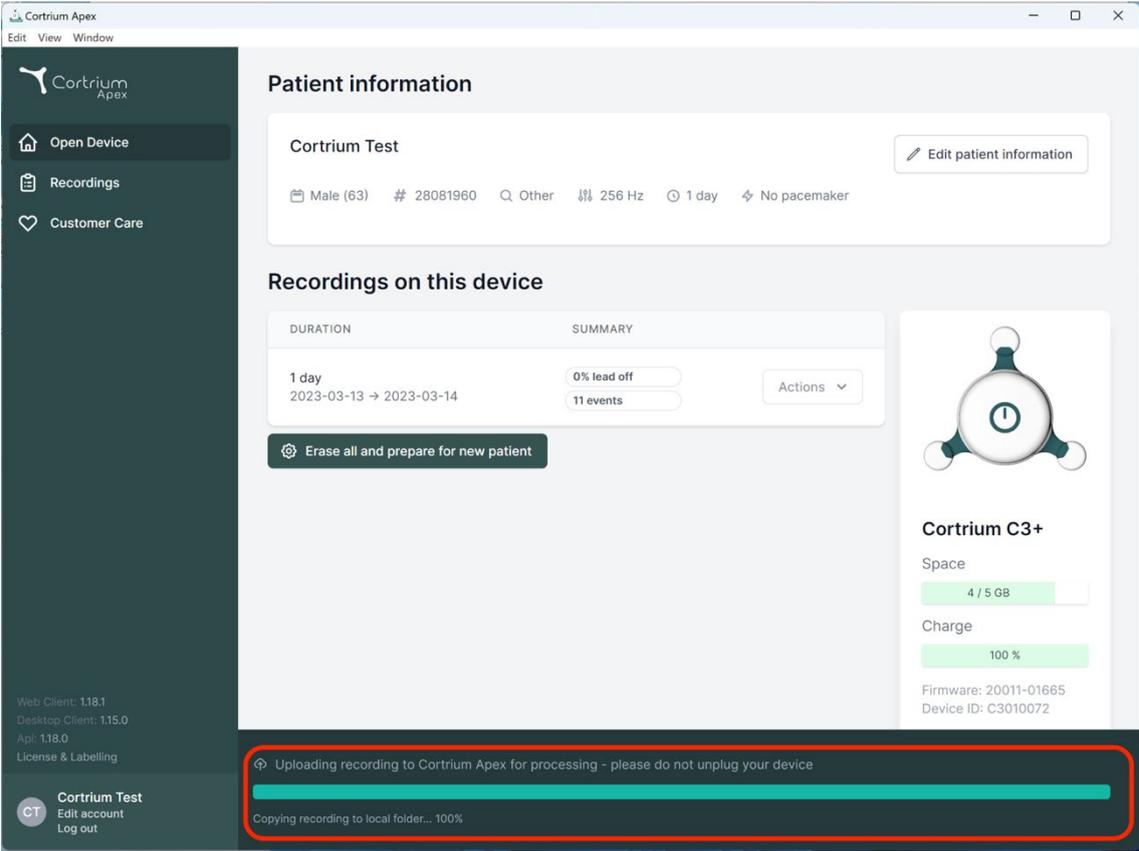Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  | Actions   |
|----------------------------------|--------------------------|-----------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | Actions ▾ |

Erase all and prepare for new patient

**Cortrium C3+**

Space: 4 / 5 GB

Charge: 100 %

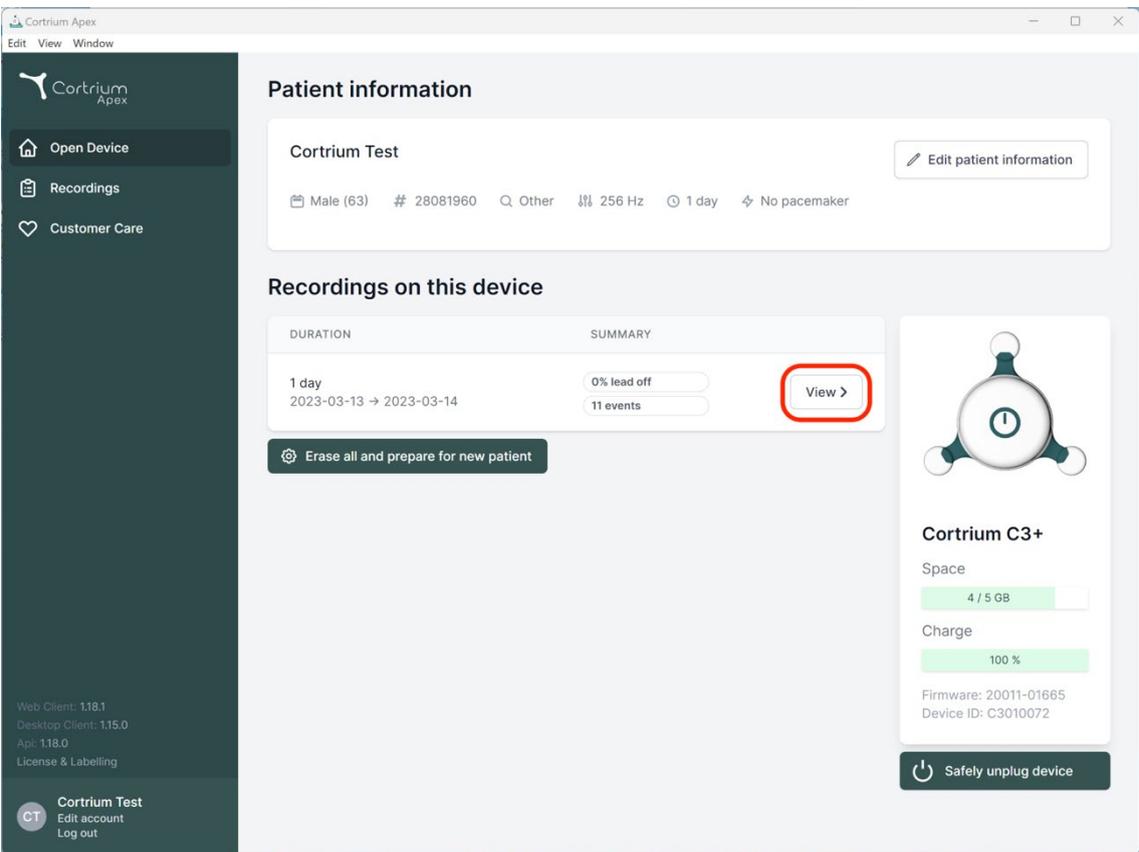
Firmware: 20011-01665  
Device ID: C3010072

Web Client: 1.18.1  
Desktop Client: 1.15.0  
Api: 1.18.0  
License & Labelling

**Cortrium Test**  
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** Edit patient information

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  | Actions          |
|----------------------------------|--------------------------|------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <b>View &gt;</b> |

Erase all and prepare for new patient

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
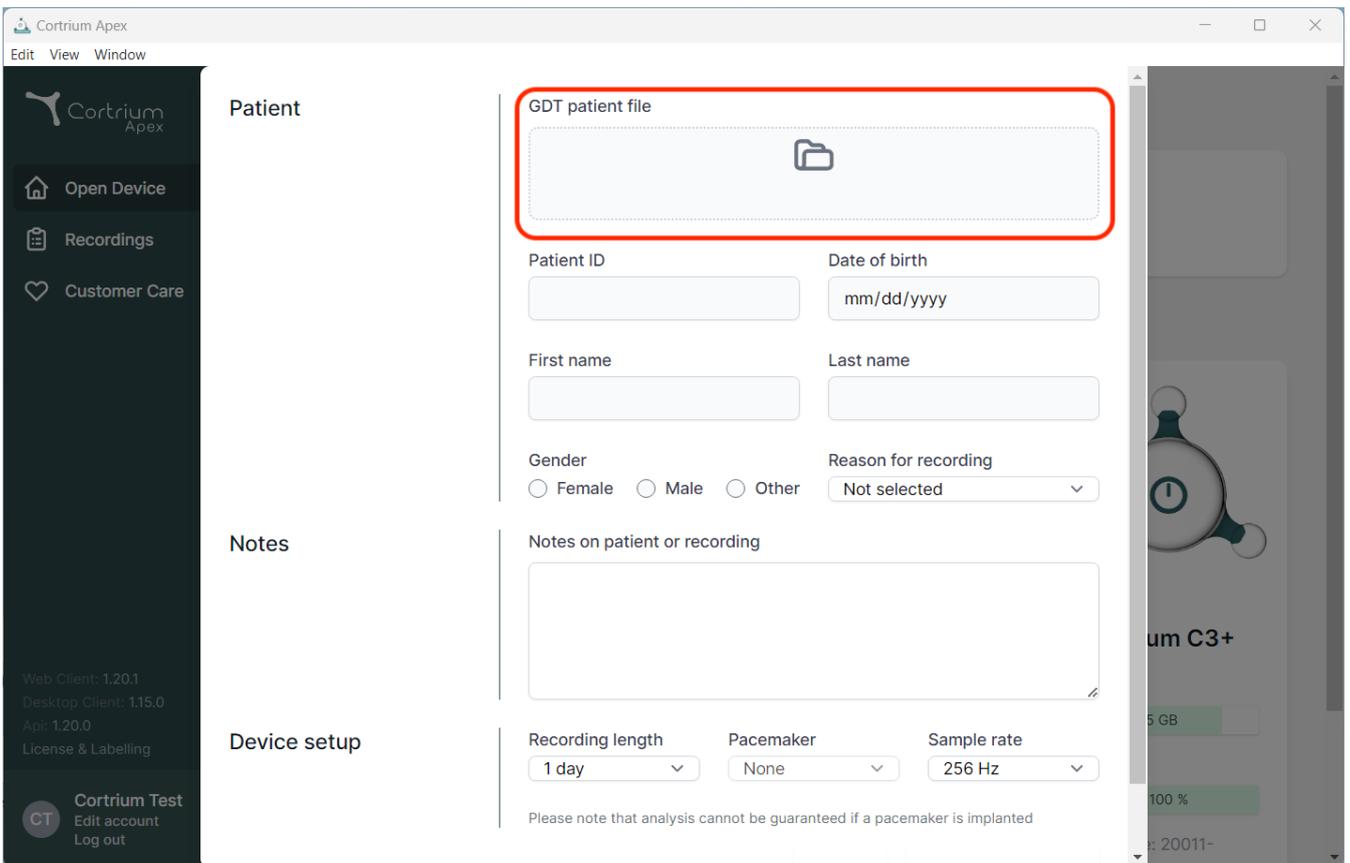
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" field, which is a file upload area. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently set to "Not selected"). There is also a "Notes" section with a text area labeled "Notes on patient or recording". At the bottom, the "Device setup" section includes "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner shows version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". The bottom right corner shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

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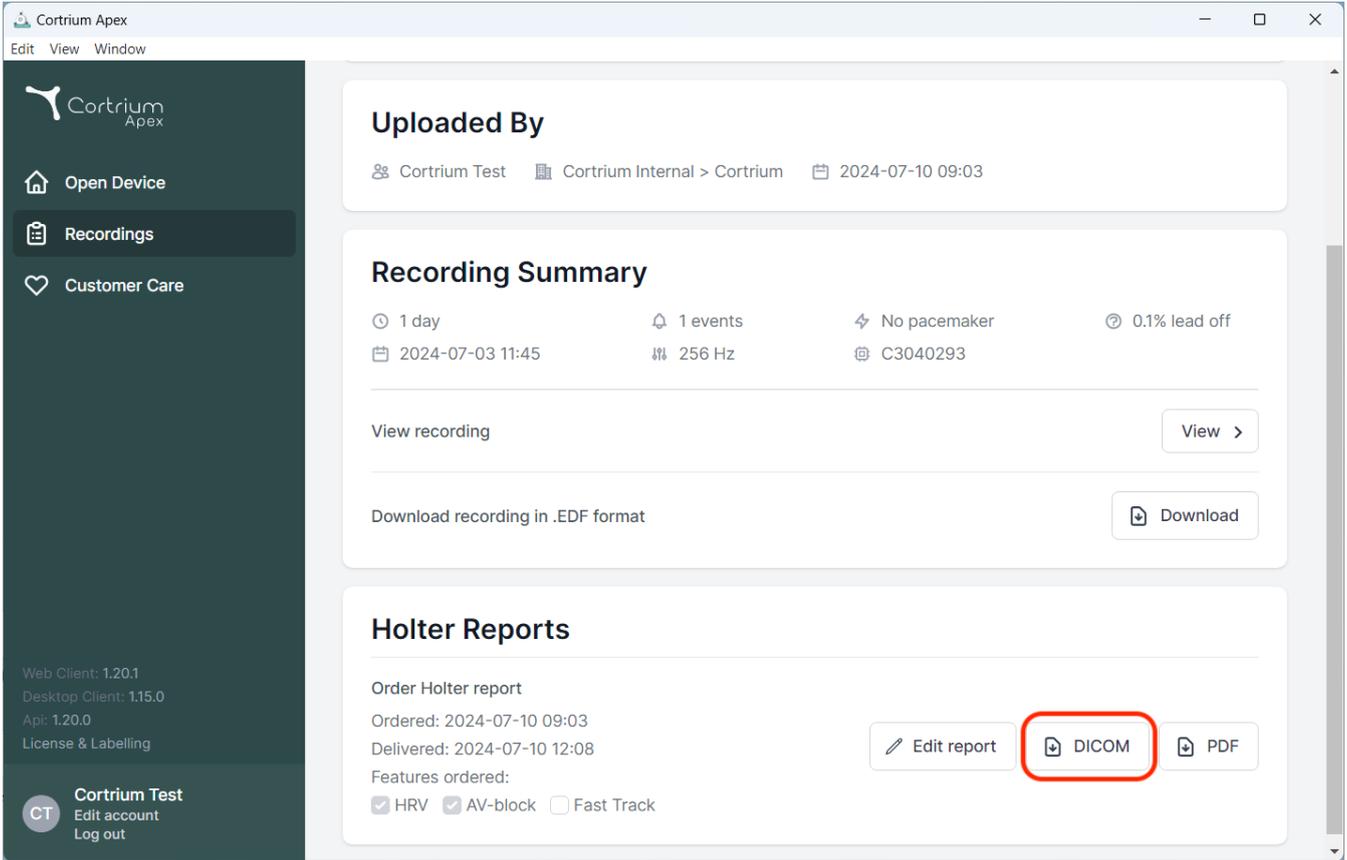
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are visible: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and 'License & Labelling' are also present.

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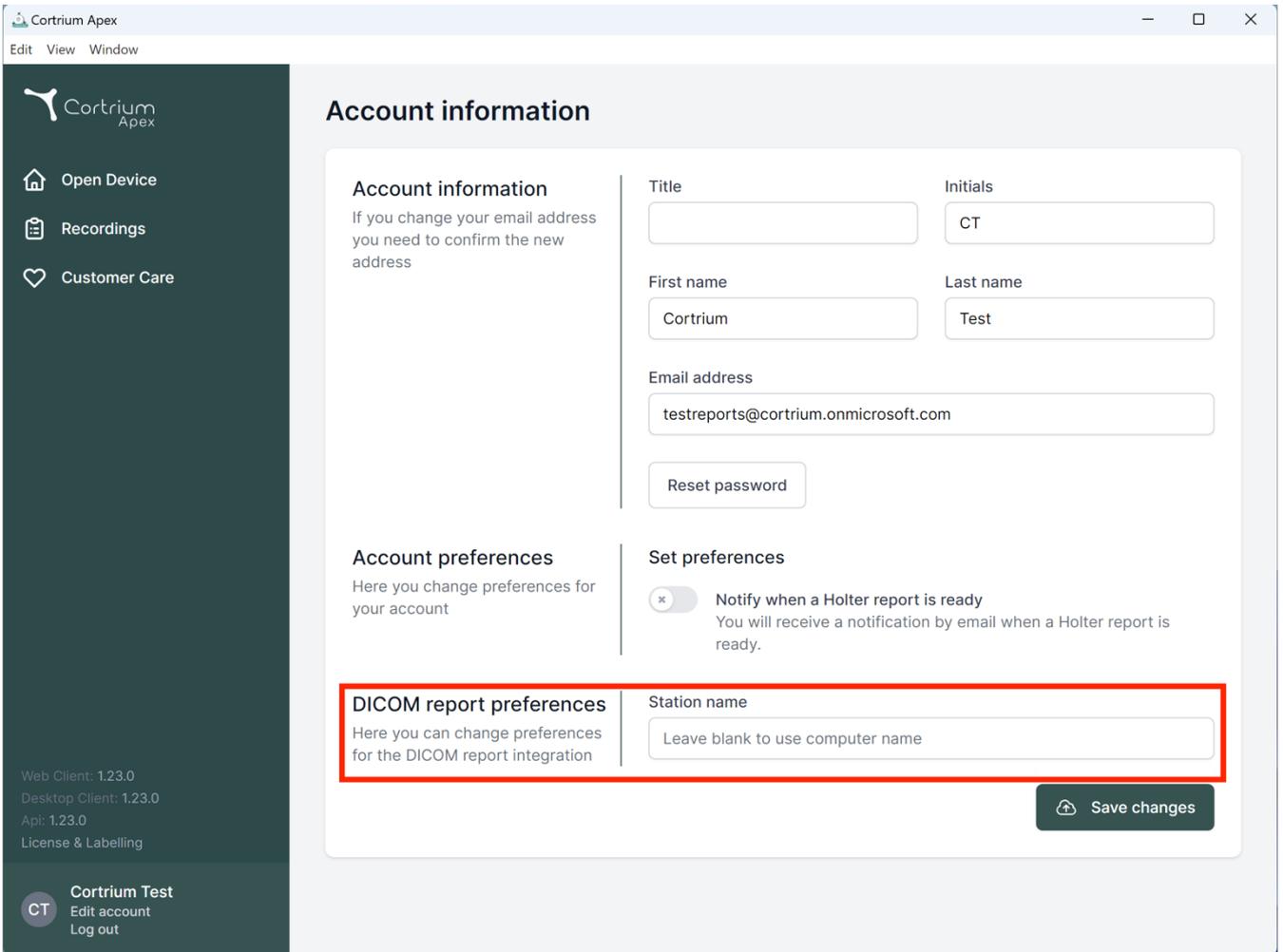**Account information**

If you change your email address you need to confirm the new address

**Account information**

Title:

Initials:

First name:

Last name:

Email address:

**Account preferences**

Here you change preferences for your account

**Set preferences**

Notify when a Holter report is ready  
You will receive a notification by email when a Holter report is ready.

**DICOM report preferences**

Here you can change preferences for the DICOM report integration

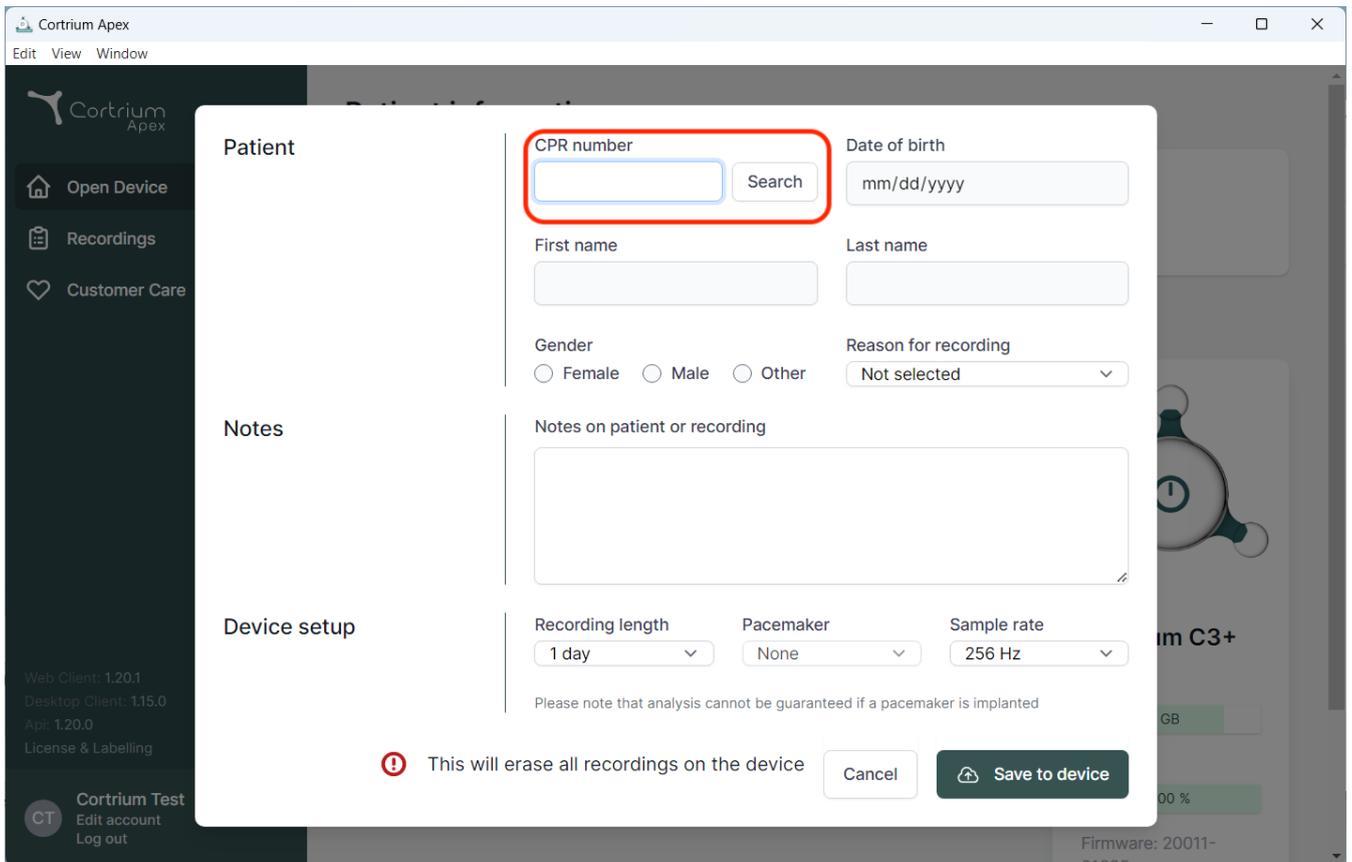
Station name:

Web Client: 1.23.0  
Desktop Client: 1.23.0  
Api: 1.23.0  
License & Labelling

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
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient search and data entry. The "CPR number" field is highlighted with a red box, and a "Search" button is located to its right. Below this, there are input fields for "First name" and "Last name", radio buttons for "Gender" (Female, Male, Other), and a dropdown menu for "Reason for recording" (currently set to "Not selected"). A "Notes" section with a text area is also present. The "Device setup" section includes dropdowns for "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". At the bottom right of the modal are "Cancel" and "Save to device" buttons.

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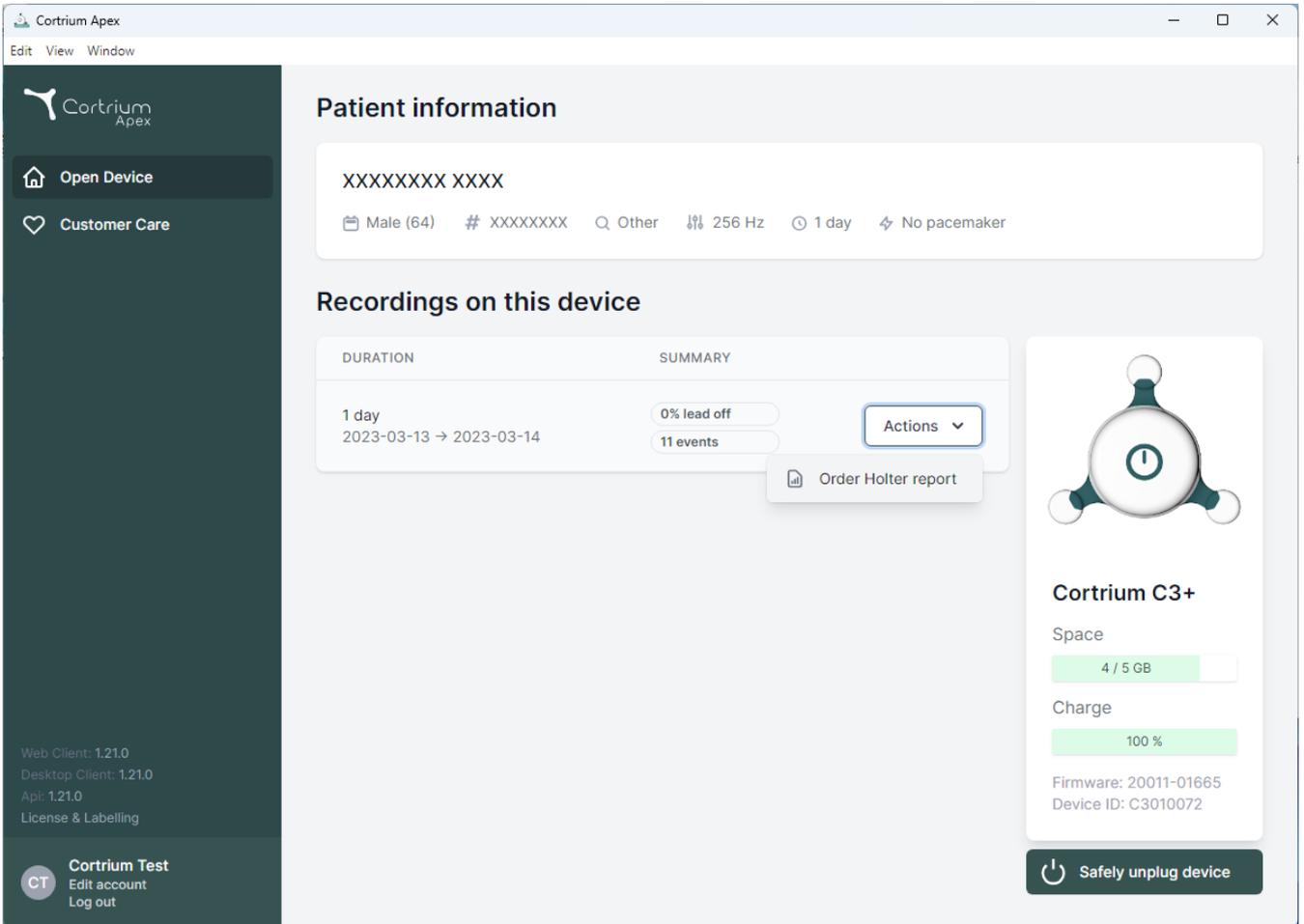
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window'. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' and 'Customer Care'. At the bottom of the sidebar, there are version details for the Web Client, Desktop Client, and API, along with a 'License & Labelling' link. A user profile section for 'Cortrium Test' is also visible, with options to 'Edit account' and 'Log out'.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with columns for 'DURATION' and 'SUMMARY'. A recording entry for '1 day' (2023-03-13 to 2023-03-14) shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, showing the option 'Order Holter report'. On the right, a device status card for 'Cortrium C3+' displays 'Space' at 4/5 GB, 'Charge' at 100%, and device information including 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of this card.

| DURATION                         | SUMMARY                  |
|----------------------------------|--------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events |

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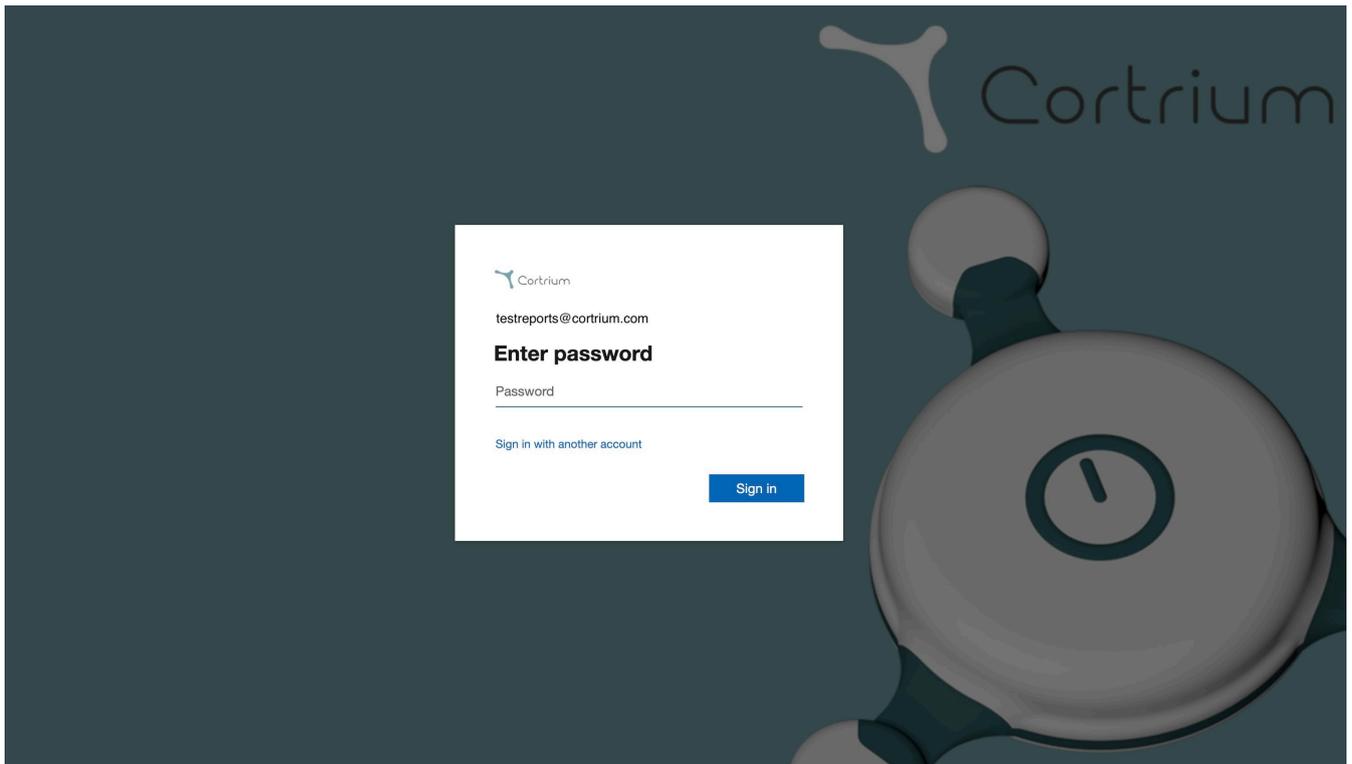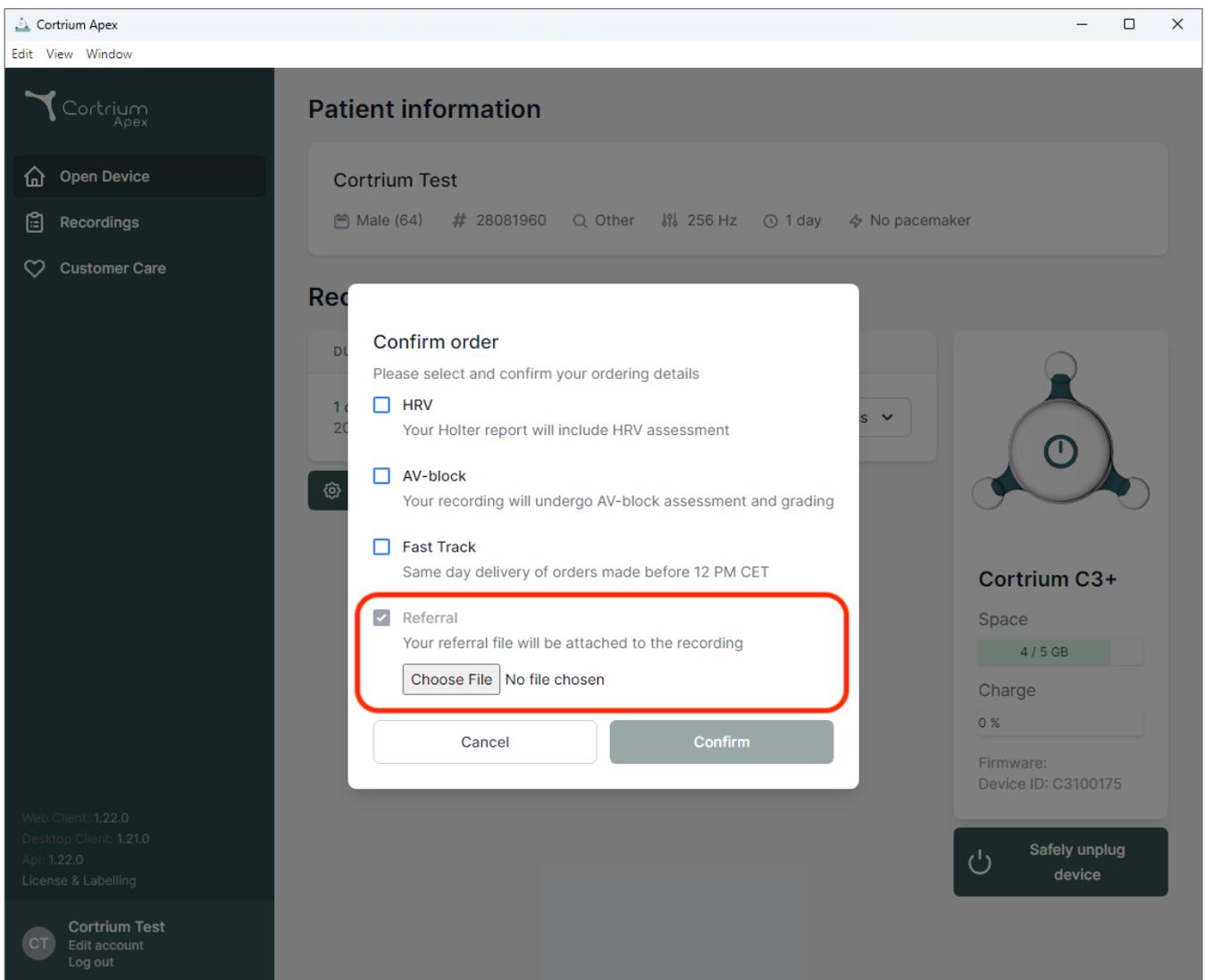
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.22.0  
Desktop Client: 1.21.0  
Api: 1.22.0  
License & Labelling

CT **Cortrium Test**  
Edit account  
Log out

### Recording details

Delete recording

#### Test Patient

Male 1950-10-01 (74) # 12345678 Afib screening

Edit

#### Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41

Referral

#### Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off  
2020-01-16 11:31 256 Hz C3020195

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
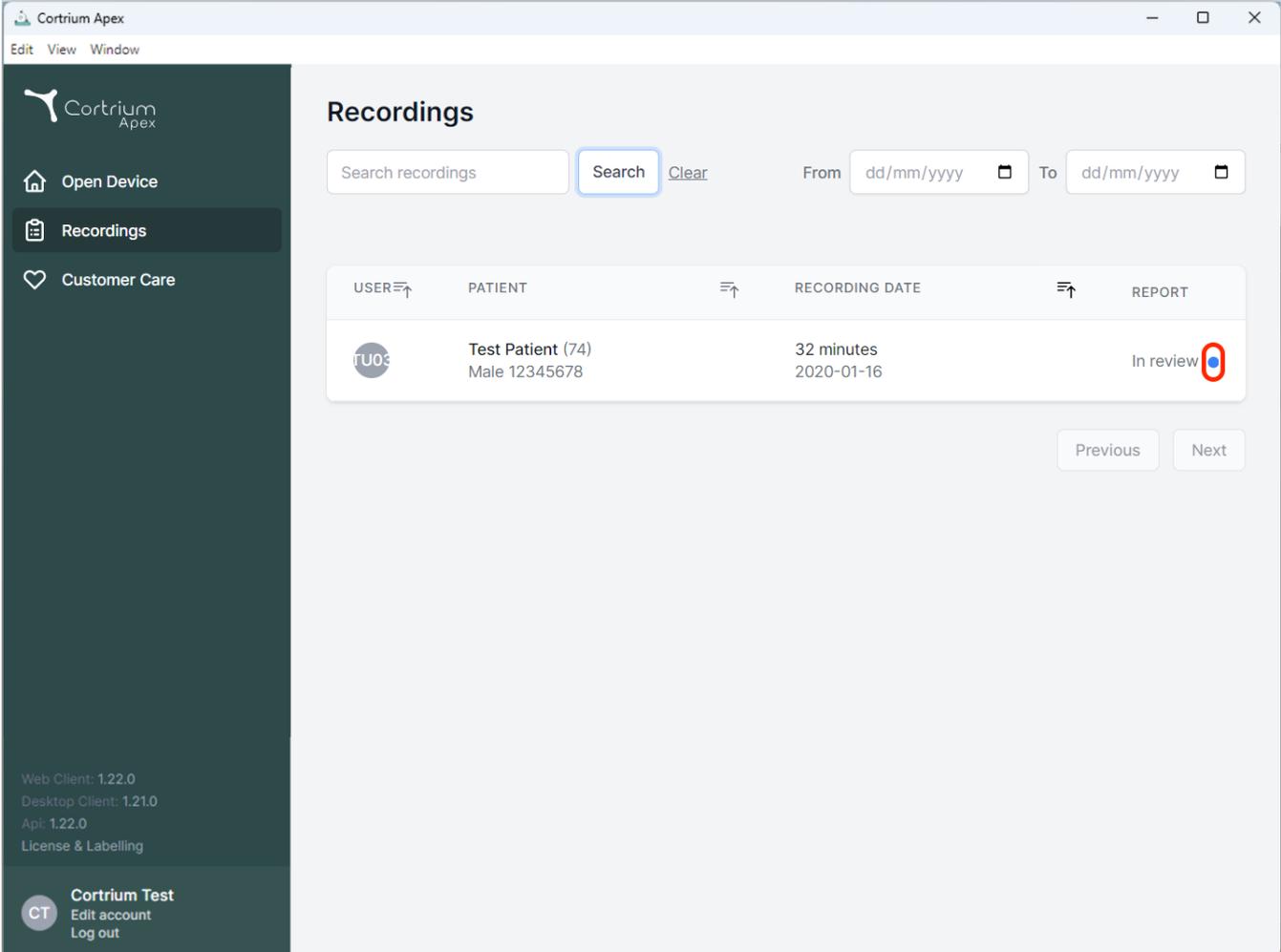
The screenshot displays the 'Recordings' view in the Cortrium Apex application. The interface includes a search bar, a date range selector, and a table of recordings. The table has the following columns: USER, PATIENT, RECORDING DATE, and REPORT. A single recording is shown with the following details:

| USER | PATIENT                            | RECORDING DATE           | REPORT  |
|------|------------------------------------|--------------------------|---|
| TU03 | Test Patient (74)<br>Male 12345678 | 32 minutes<br>2020-01-16 | In review  |

At the bottom of the table, there are 'Previous' and 'Next' navigation buttons. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The bottom of the sidebar shows the user's name 'Cortrium Test' and options to edit the account or log out. The footer of the sidebar lists version information: Web Client: 1.22.0, Desktop Client: 1.21.0, and Api: 1.22.0.

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
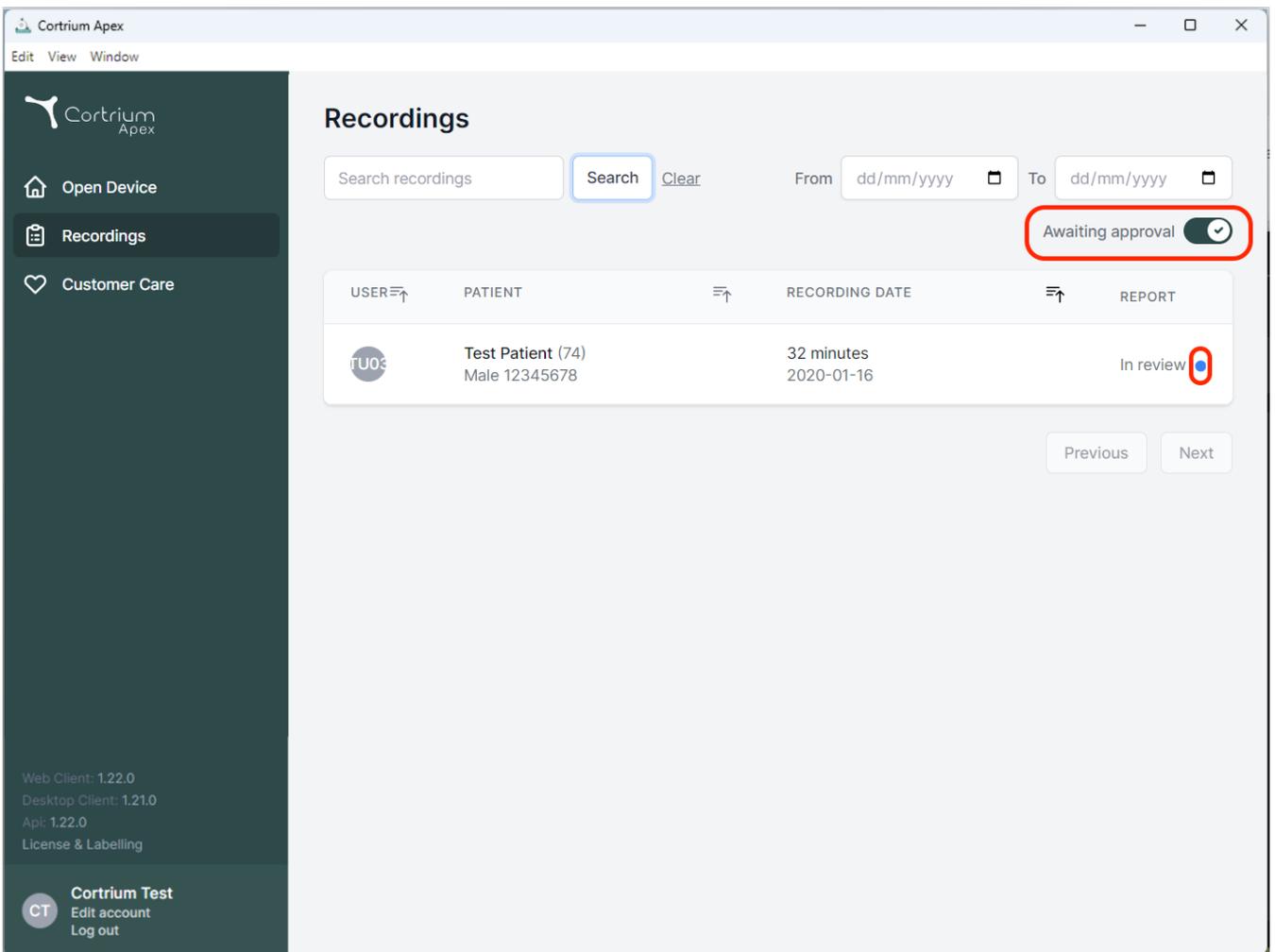
The screenshot shows the Cortrium Apex web application interface. The sidebar on the left contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Recordings' and features a search bar, a 'Search' button, and a 'Clear' button. Below the search bar, there are date pickers for 'From' and 'To' (dd/mm/yyyy). A red box highlights the 'Awaiting approval' toggle switch, which is currently turned on. Below this, there is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one row with the following data: USER: TU03, PATIENT: Test Patient (74) Male 12345678, RECORDING DATE: 32 minutes 2020-01-16, REPORT: In review. A red box highlights a blue dot next to the 'In review' status. At the bottom of the table, there are 'Previous' and 'Next' buttons. The footer of the application shows version information: Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling, and user information: Cortrium Test, Edit account, Log out.

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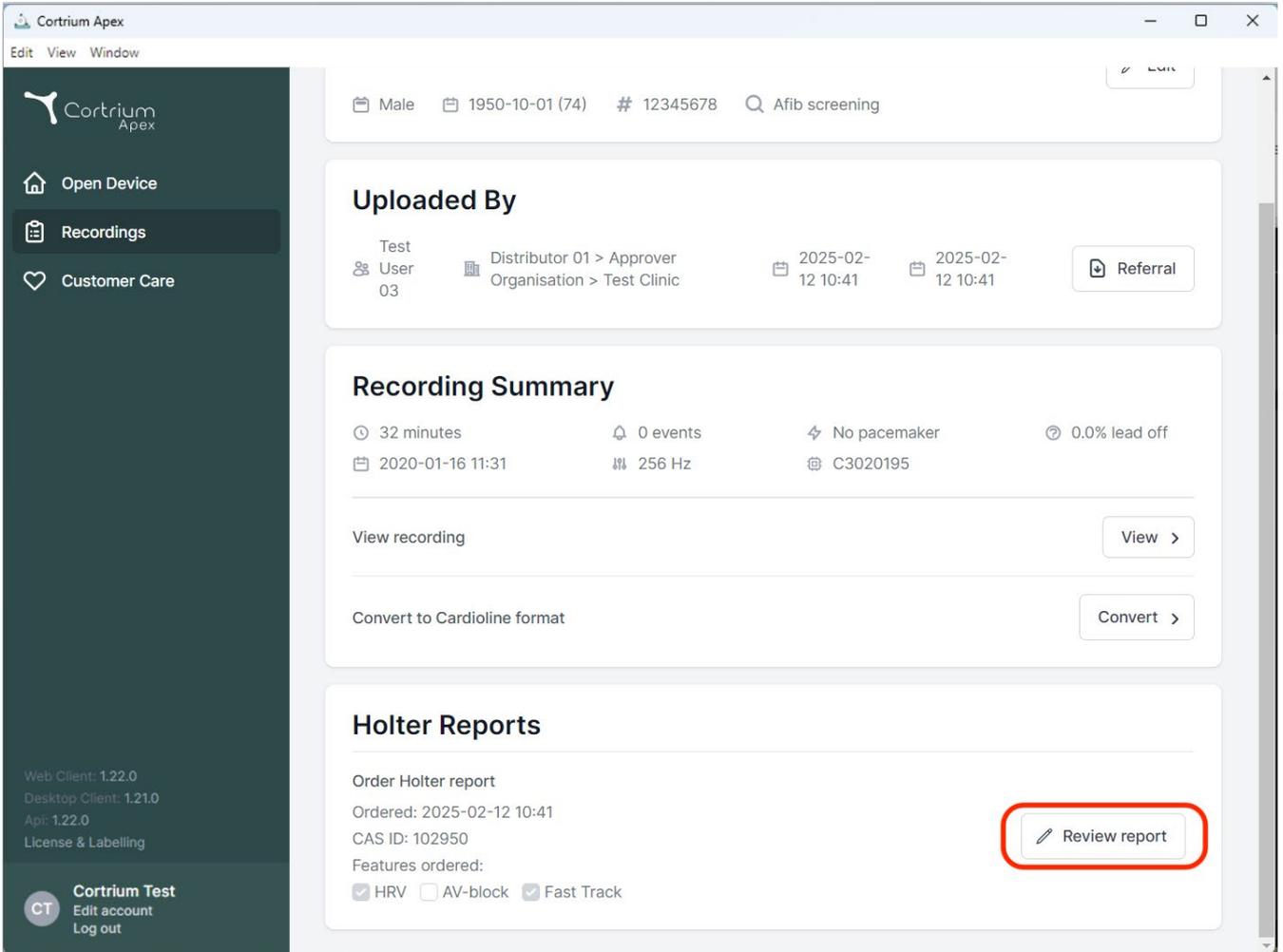The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the 'Uploaded By' section, indicating the test was performed by User 03 on 2025-02-12 10:41. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, recorded on 2020-01-16 11:31 at 256 Hz, with ID C3020195. Action buttons for 'View recording' and 'Convert to Cardioline format' are present. The 'Holter Reports' section shows an order for a Holter report on 2025-02-12 10:41, with CAS ID 102950. The 'Features ordered' section includes HRV (checked), AV-block (unchecked), and Fast Track (checked). A red box highlights the 'Review report' button in the bottom right corner of the Holter Reports section.

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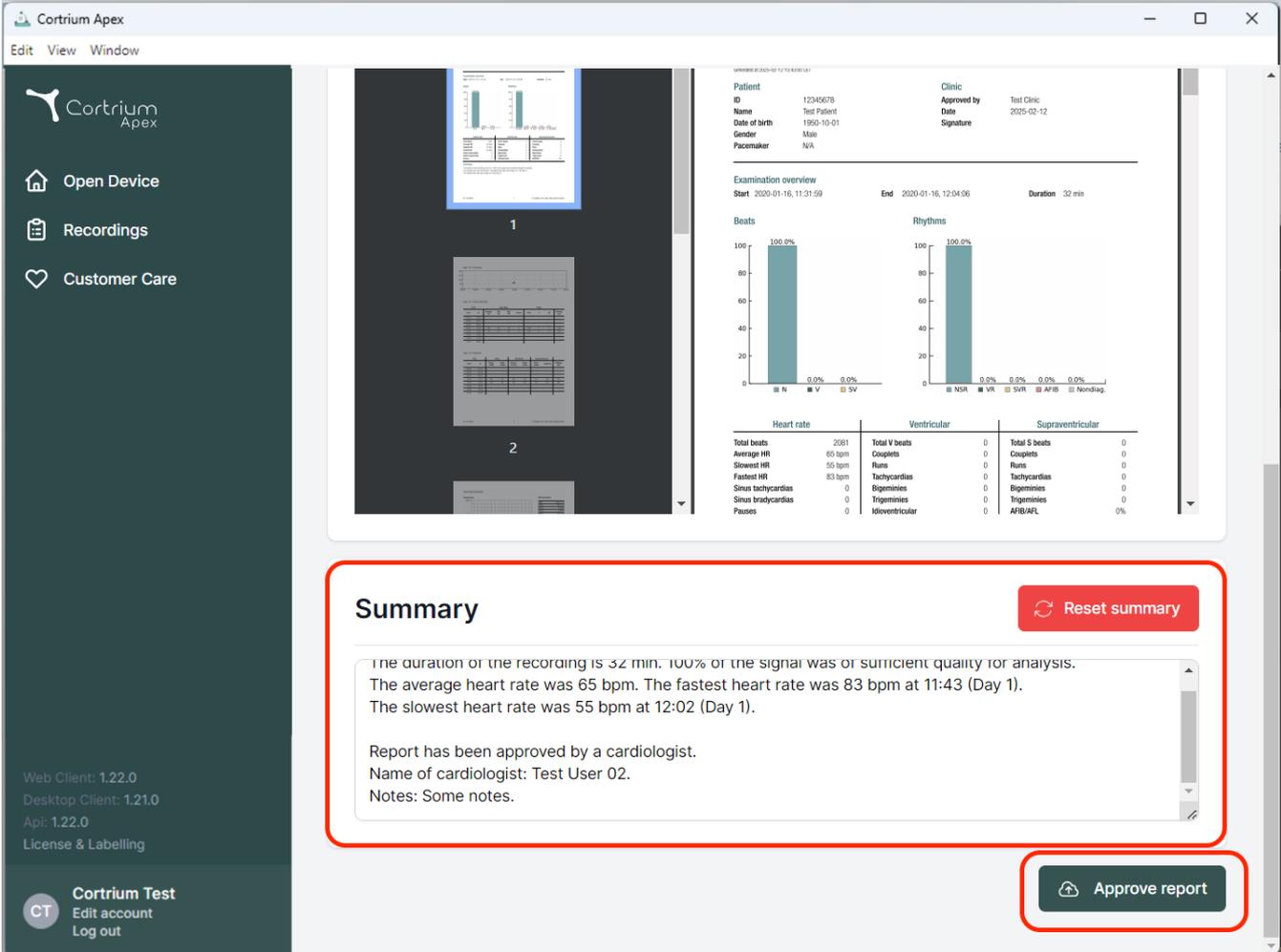The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area shows a patient report for 'Test Patient' (ID: 12345678, Date of birth: 1950-10-01, Gender: Male, Pacemaker: N/A). The report includes an examination overview (Start: 2020-01-16, 11:31:59; End: 2020-01-16, 12:04:06; Duration: 32 min) and two bar charts: 'Beats' (100.0% NSR, 0.0% V, 0.0% SV) and 'Rhythms' (100.0% NSR, 0.0% VR, 0.0% SVR, 0.0% AFIB, 0.0% Nondiag). Below the charts is a table with columns for Heart rate, Ventricular, and Supraventricular metrics.

The 'Summary' section is highlighted with a red border and contains the following text:

**Summary** Reset summary


The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.  
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).  
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.  
 Name of cardiologist: Test User 02.  
 Notes: Some notes.

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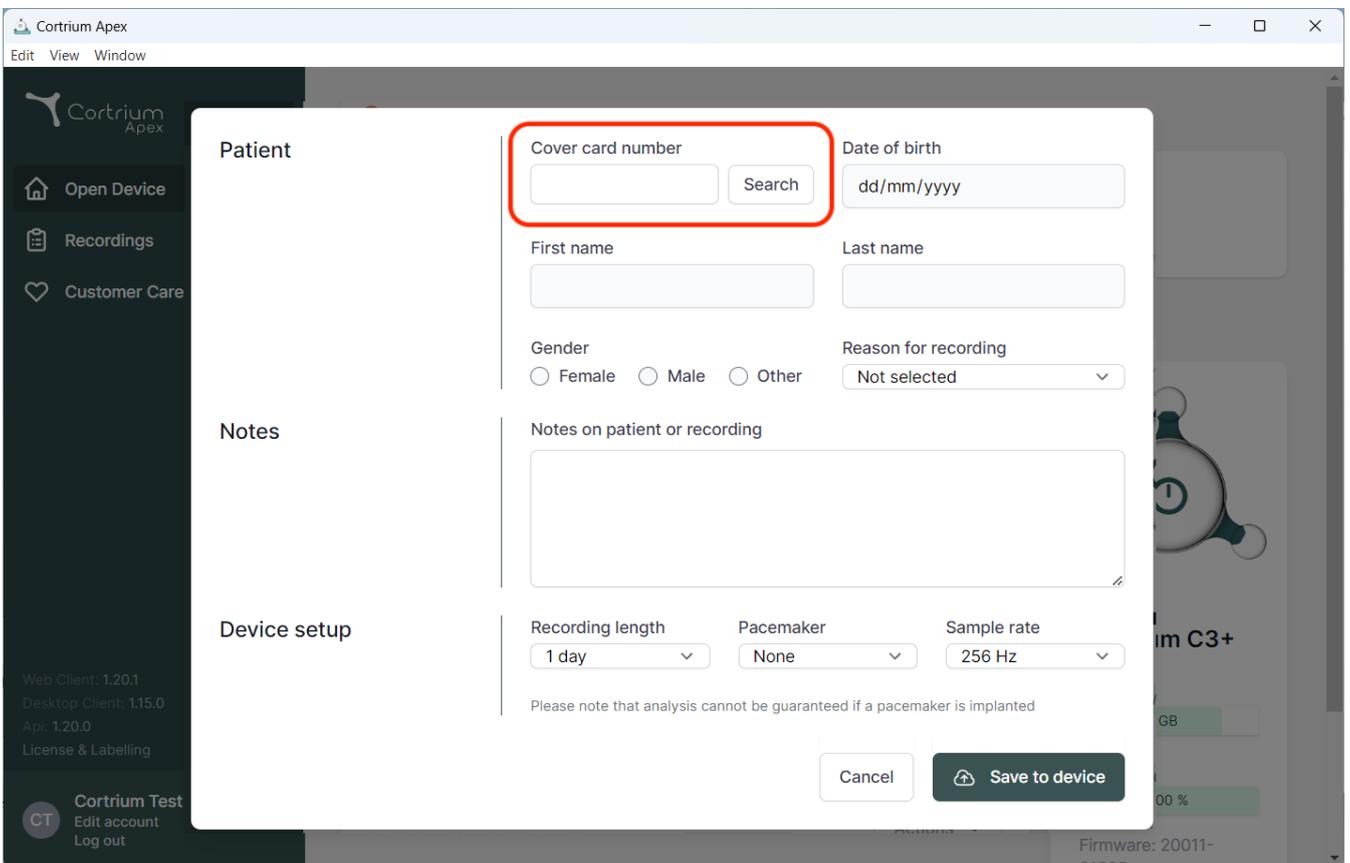The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient search and device configuration. The "Cover card number" field is highlighted with a red box, and a "Search" button is located next to it. The form includes fields for "Date of birth" (format: dd/mm/yyyy), "First name", and "Last name". There are radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected"). A text area for "Notes on patient or recording" is also present. Under the "Device setup" section, there are dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom of the modal, there are "Cancel" and "Save to device" buttons. The background shows the main application menu with options like "Open Device", "Recordings", and "Customer Care", along with version information and user account details.

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark  
E-mail : [info@cortrium.com](mailto:info@cortrium.com) • Site Web : [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033

**CE**  
**0123**  
48 / 48