Cortrium Apex 1.26

Cortrium Apex 1.26

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
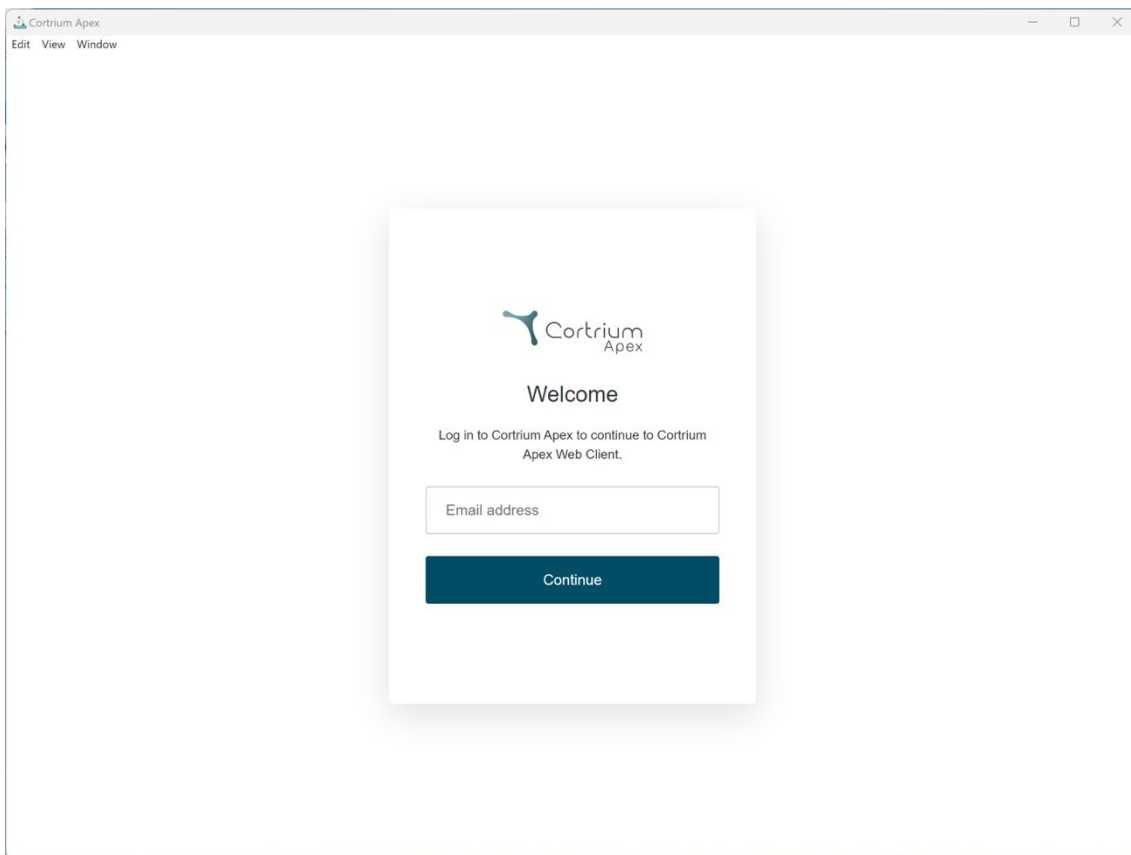
The screenshot shows a web browser window titled "Cortrium Apex". The browser's menu bar includes "Edit", "View", and "Window". In the center of the browser window is a white login card with a subtle shadow. The card features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" below it.

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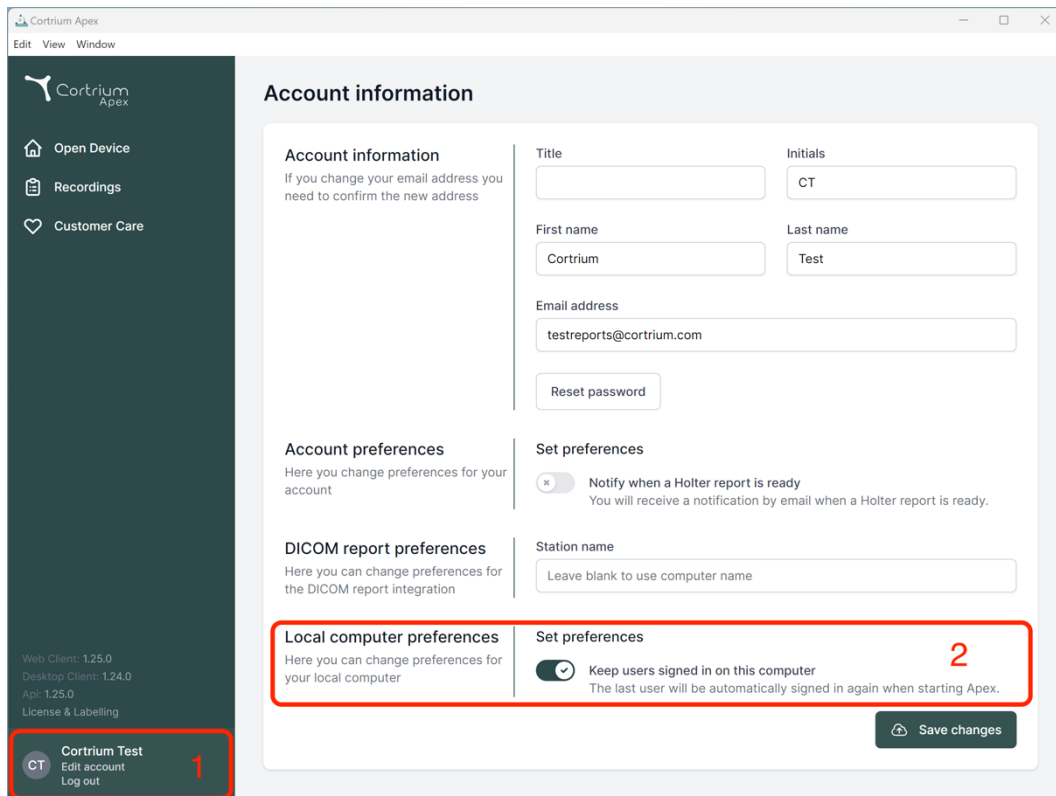The screenshot displays the Cortrium Apex application window. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details for Web Client, Desktop Client, and API. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with a red box labeled '1' around it. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' with a toggle for notifications; 'DICOM report preferences' with a station name field; and 'Local computer preferences' with a toggle for keeping users signed in, which is highlighted with a red box labeled '2'. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

Account preferences

Here you change preferences for your account

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DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

Local computer preferences

Here you can change preferences for your local computer

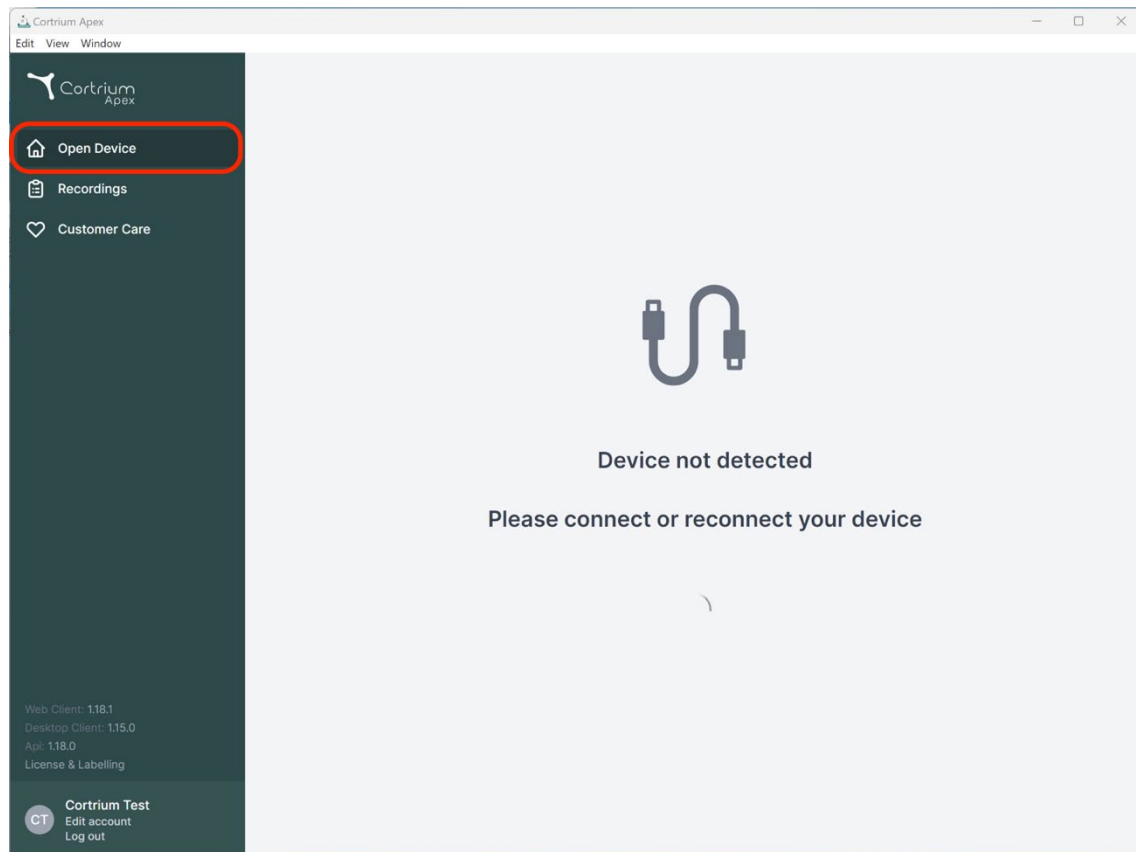
☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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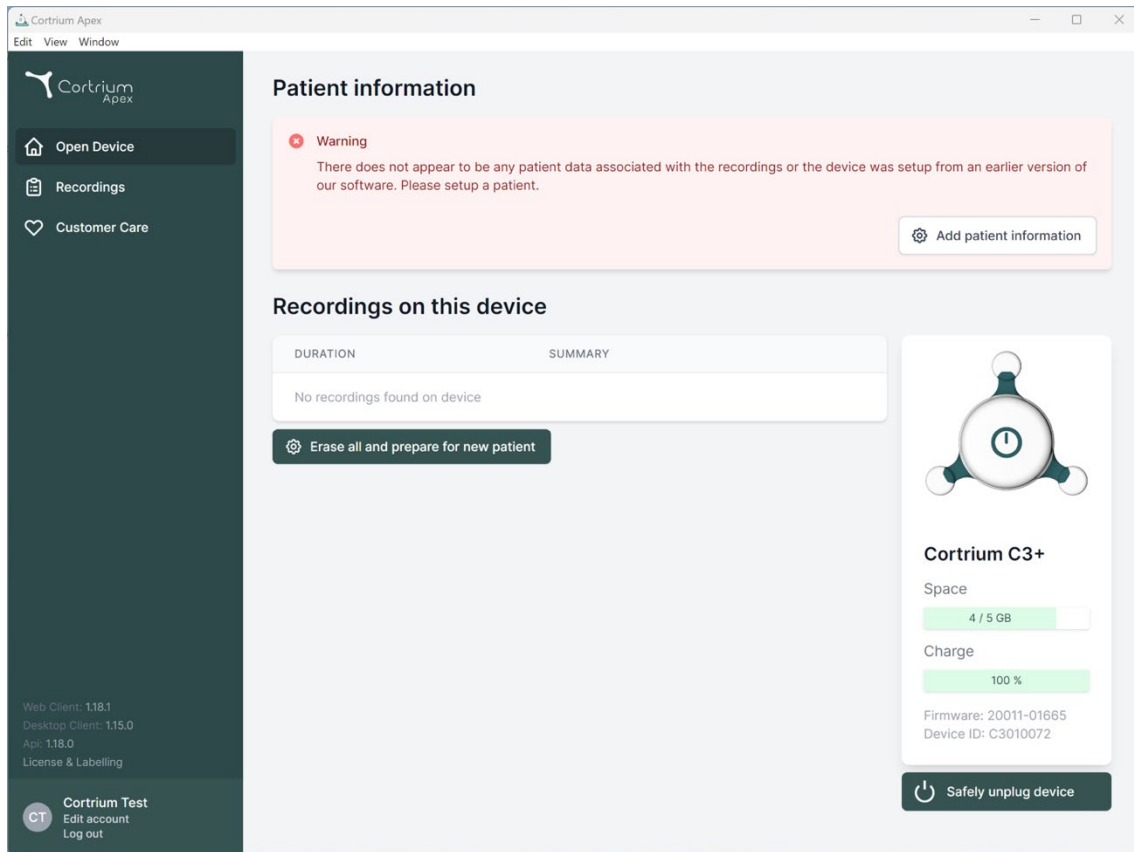
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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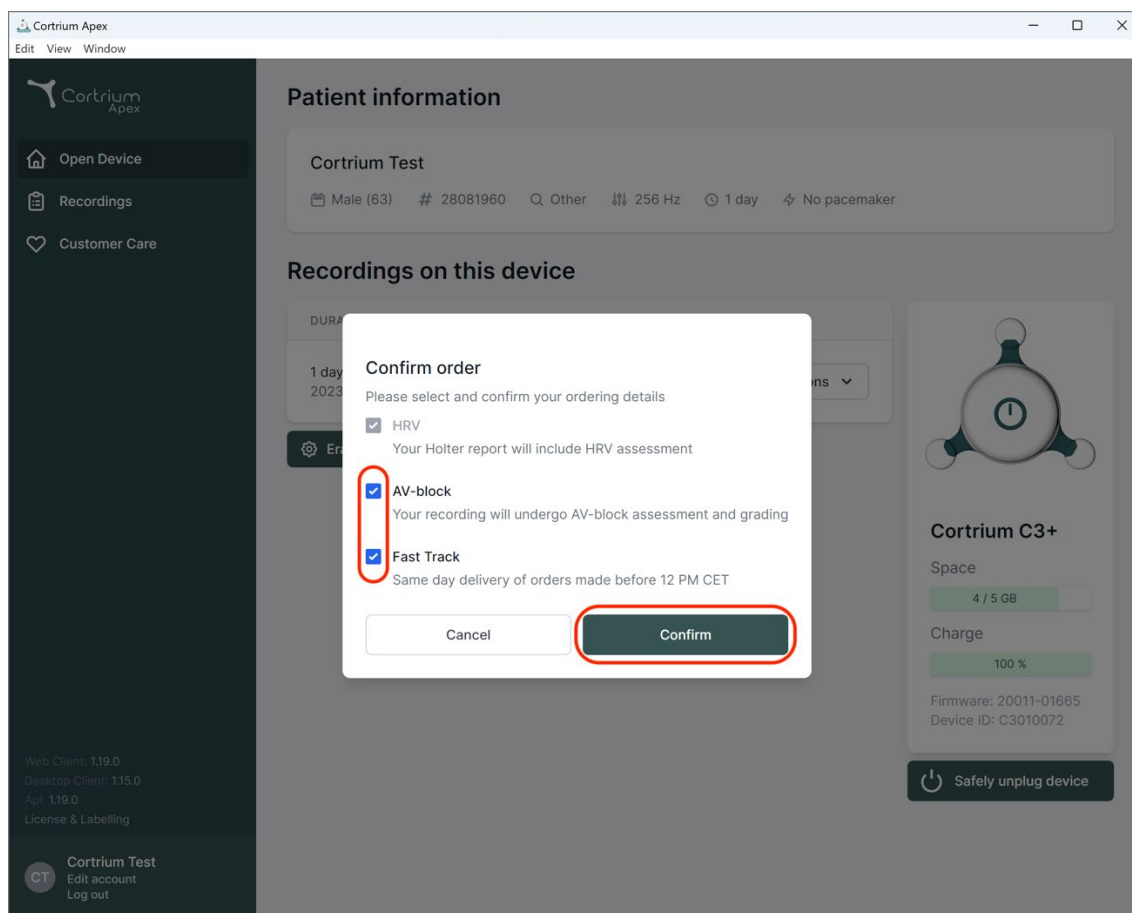
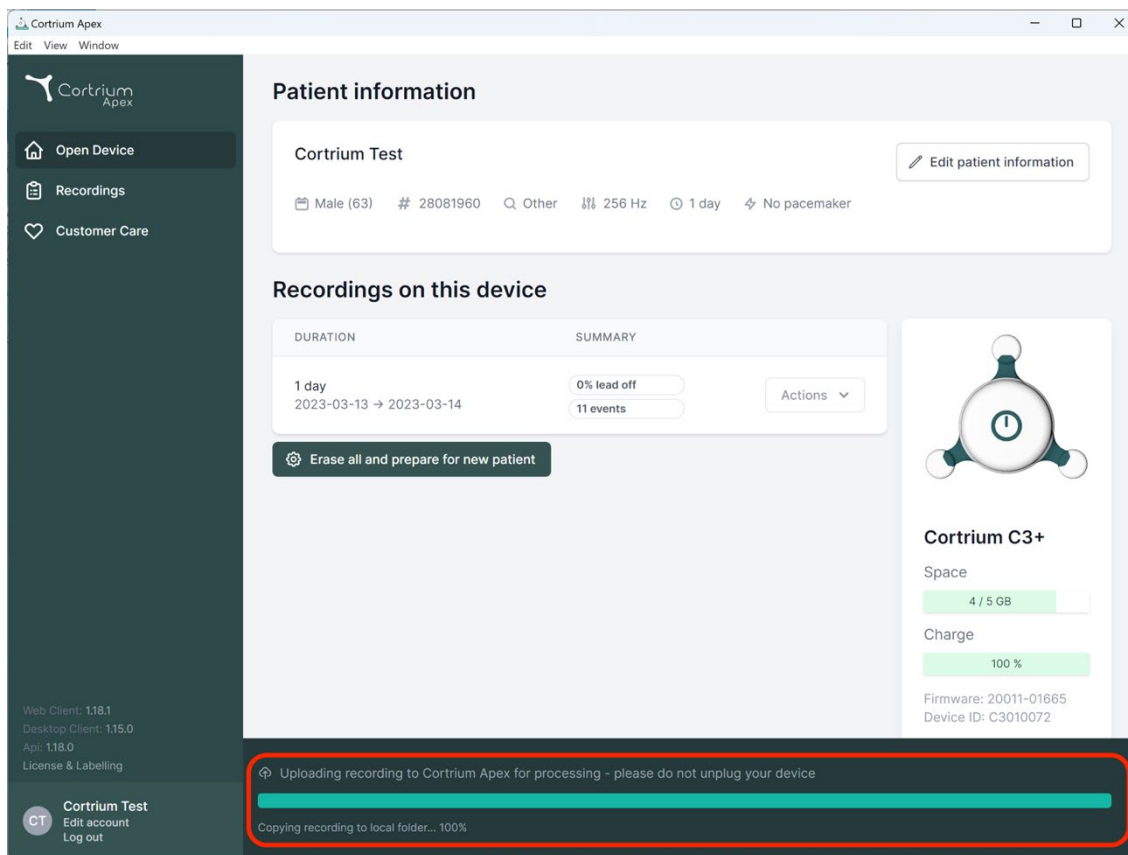
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into sections: Patient information, Recordings on this device, and device status for Cortrium C3+.

Patient information

Cortrium Test

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Edit patient information

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

Erase all and prepare for new patient

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

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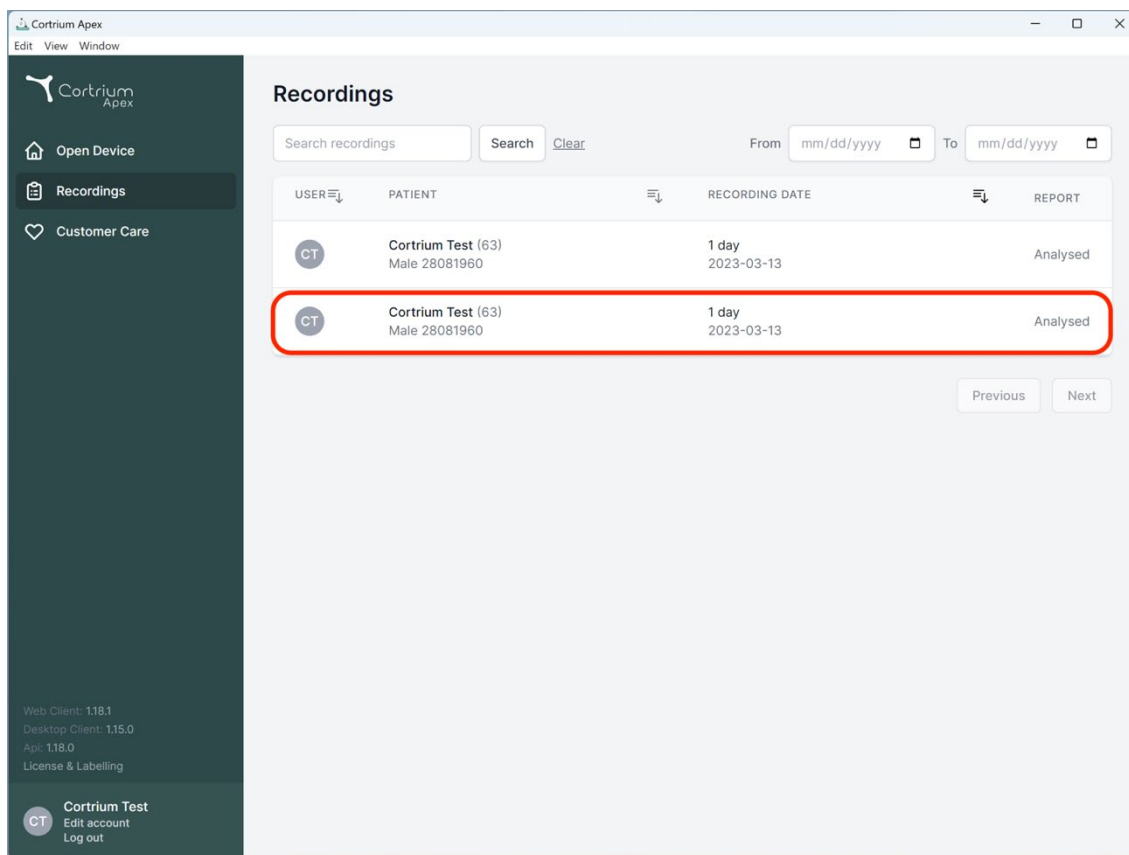
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area features a search bar and a table of recordings. The table has columns for 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. Two rows are visible, both for 'Cortrium Test (63)' with patient ID 'Male 28081960' and a recording date of '1 day 2023-03-13'. The 'REPORT' column for both rows indicates 'Analysed'. The second row is highlighted with a red border. Below the table are 'Previous' and 'Next' navigation buttons. The sidebar also contains version information and a user profile section for 'Cortrium Test'.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex

Edit View Window

[Open Device](#)

[Recordings](#)

[Customer Care](#)

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT

Cortrium Test

Edit account
Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 2023-03-13 11:54

11 events 256 Hz

No pacemaker C3100175

0.0% lead off

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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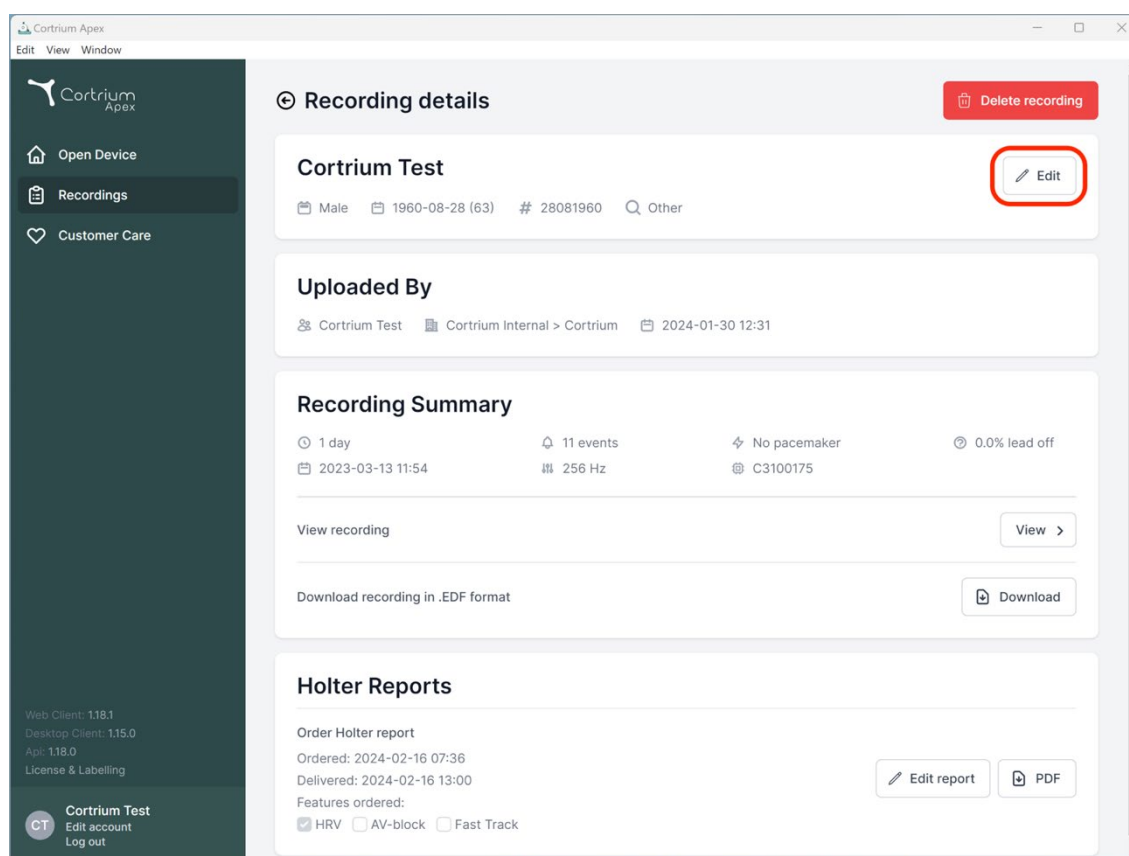
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

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
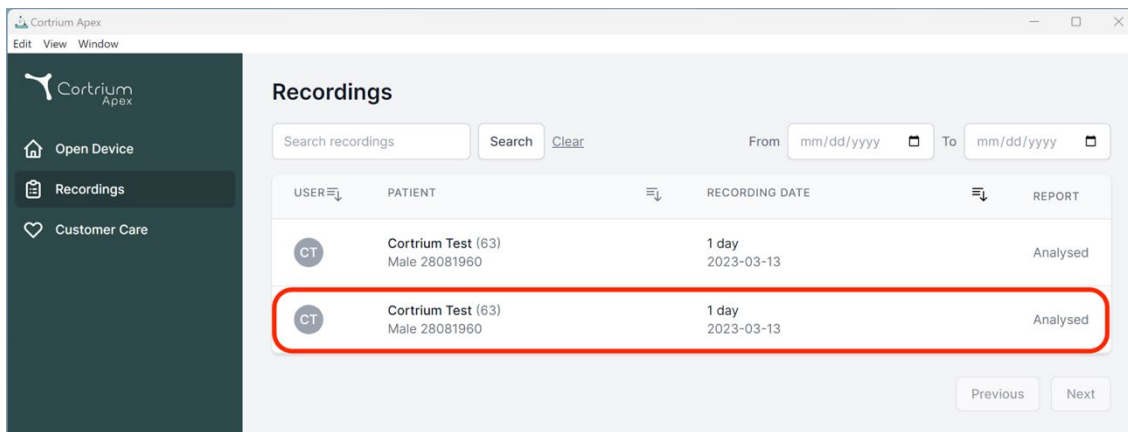
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. It features a search bar, a date range selector, and a table of recordings. The table has columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red rectangle.

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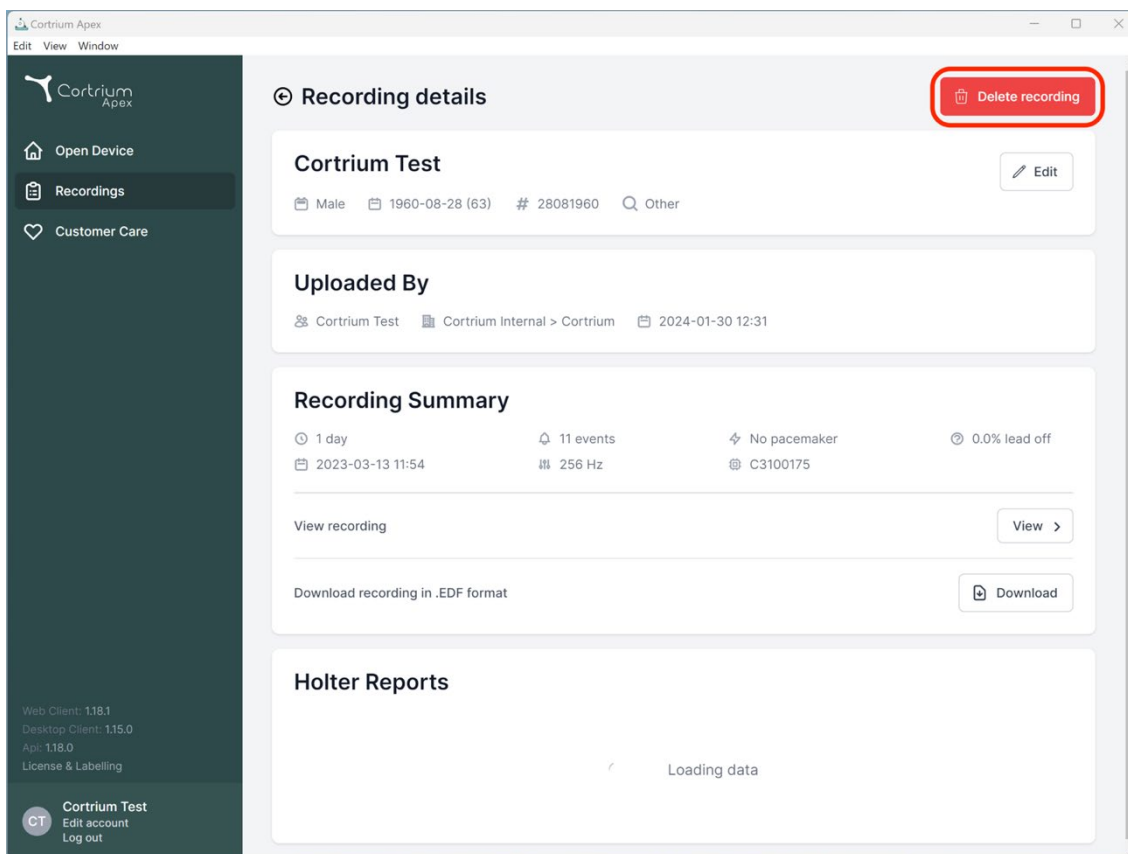The screenshot shows the 'Recording details' page for 'Cortrium Test'. It includes a 'Delete recording' button in the top right corner, which is highlighted with a red rectangle. The page displays patient information, upload details, a recording summary, and a section for Holter reports.

Recording details

Cortrium Test [Edit]

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View >]

Download recording in .EDF format [Download]

Holter Reports

Loading data

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 118.1

Desktop Client: 115.0

App: 118.0

License & Labelling

CT Cortrium Test

Edit account

Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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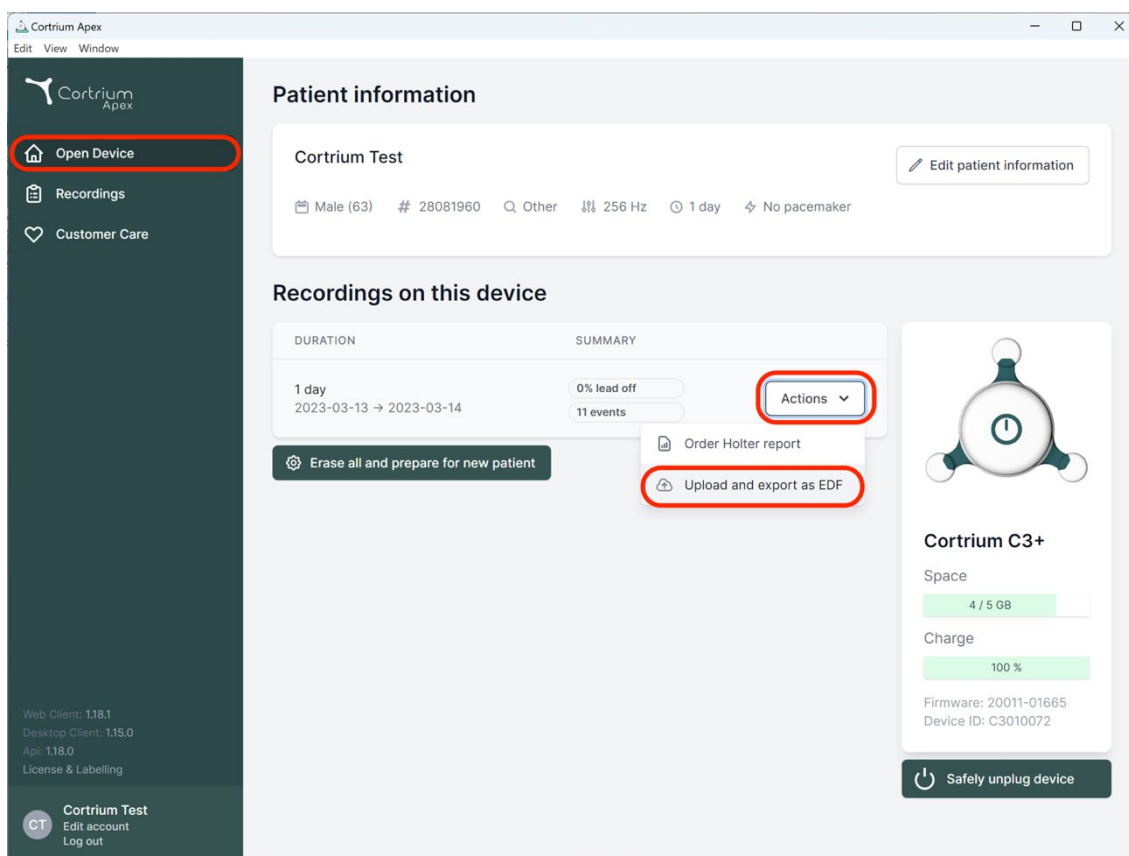
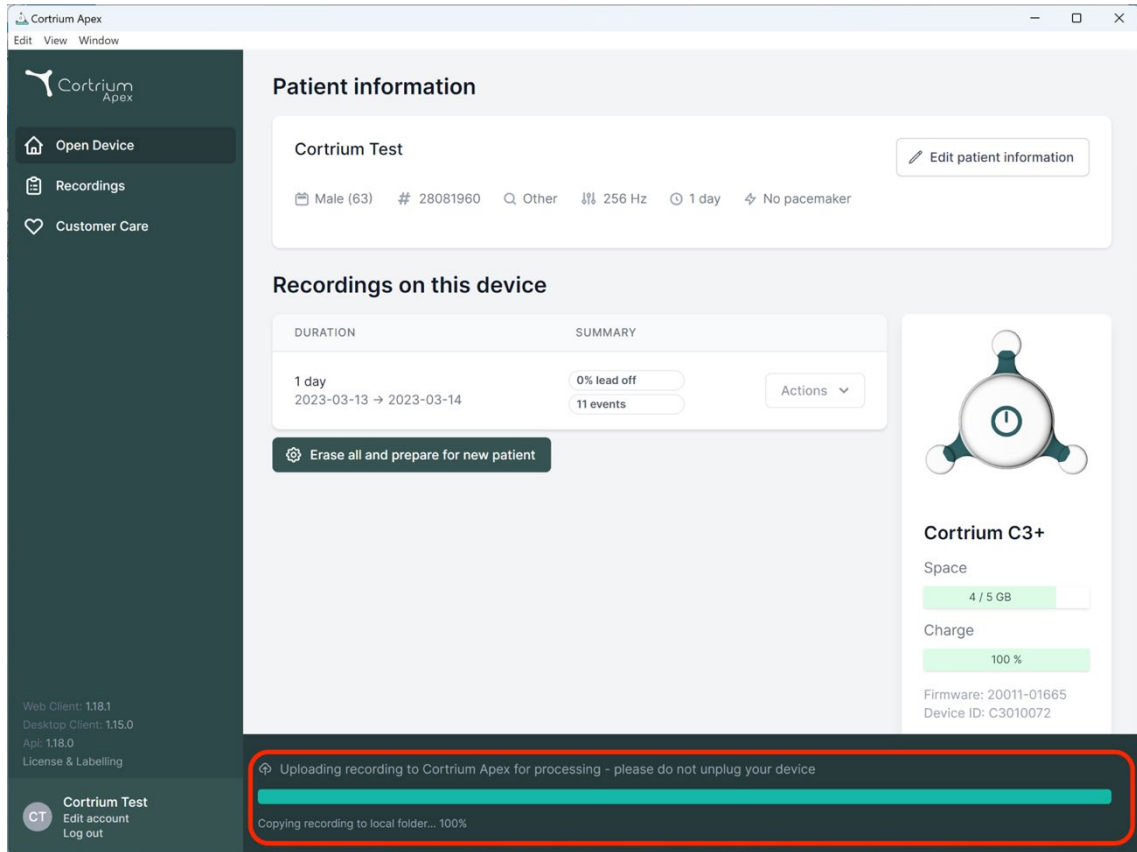
[Erase all and prepare for new patient](#)

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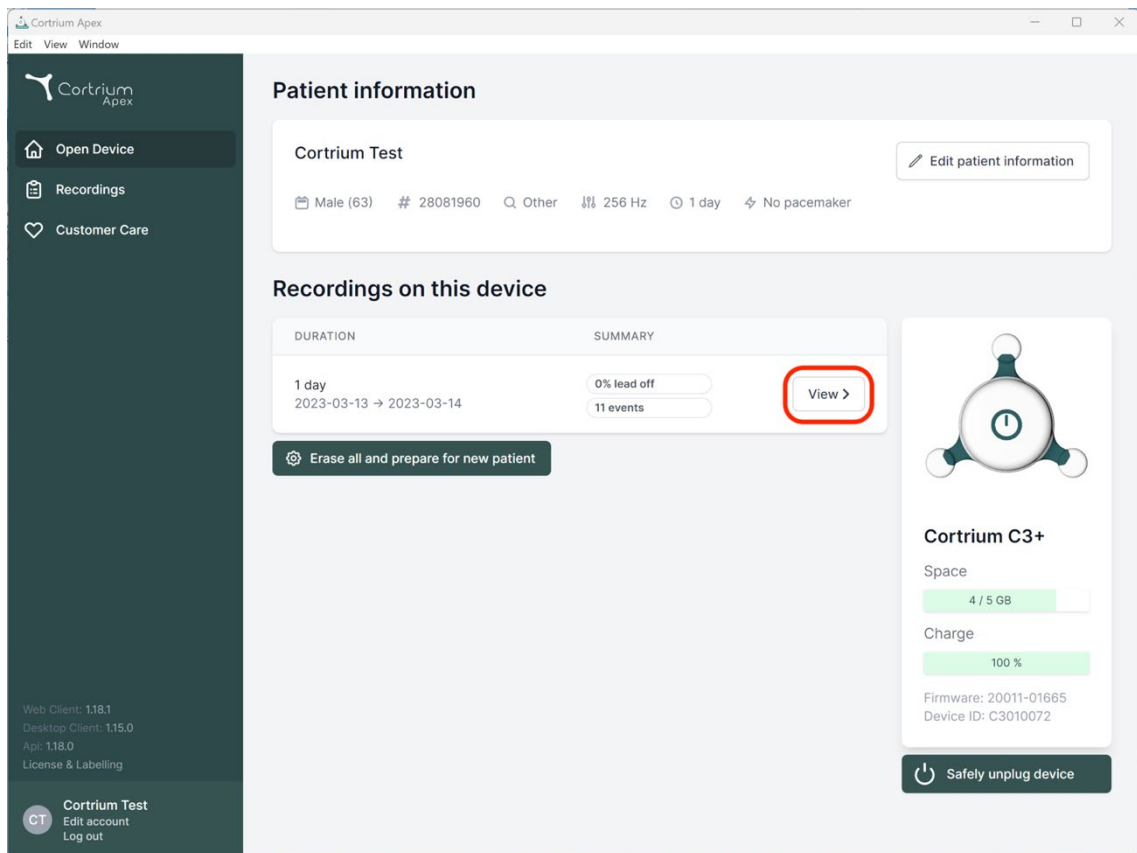
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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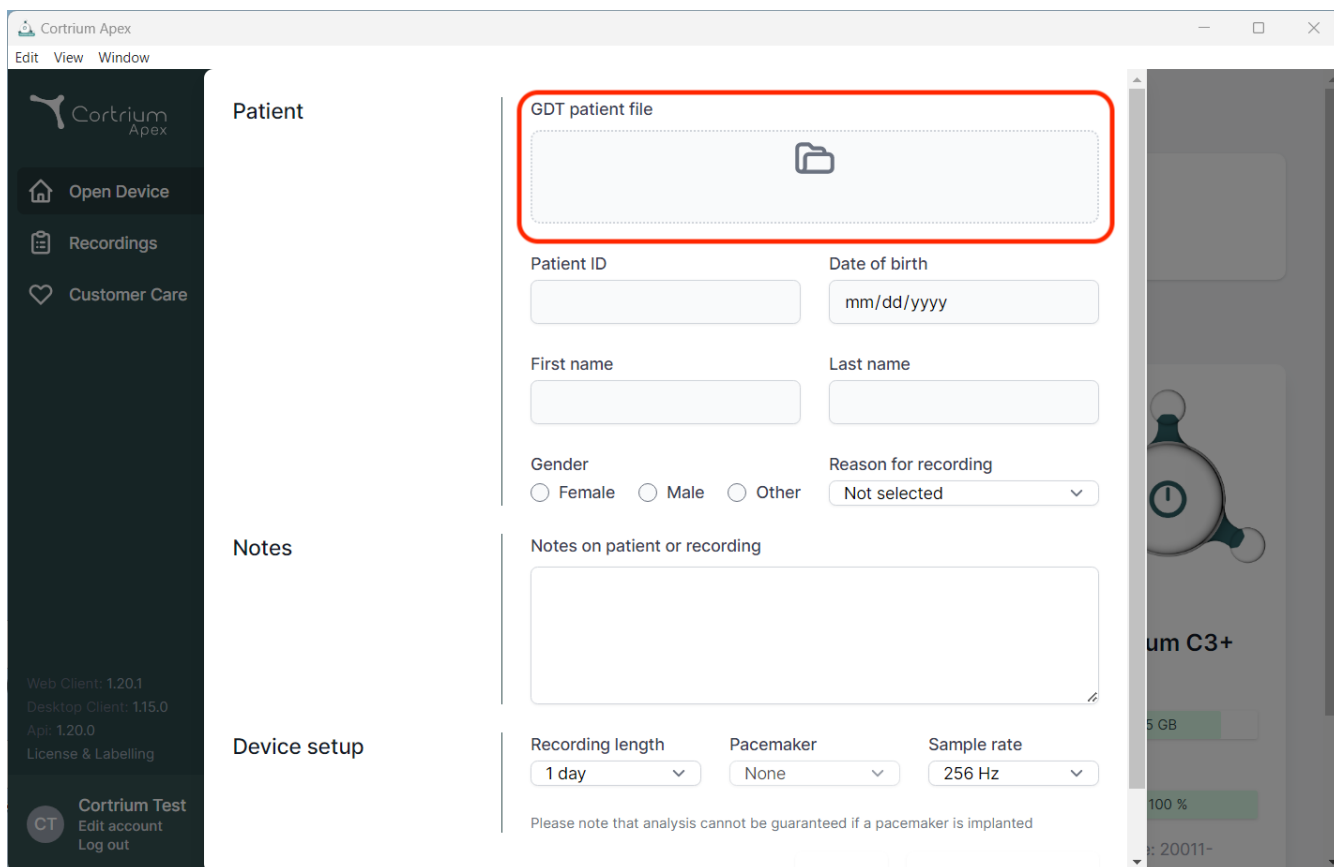
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

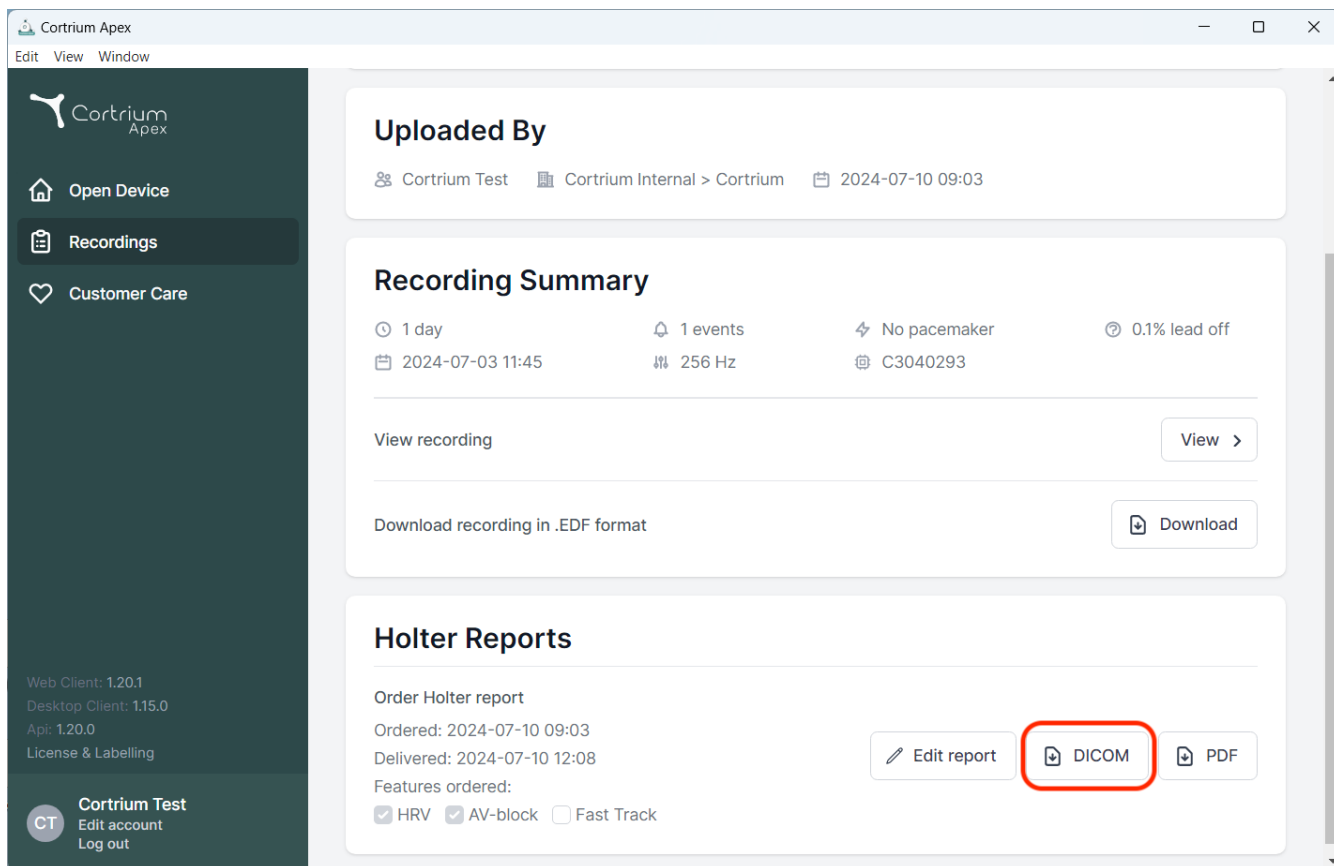
The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

- Patient section:** Contains a "GDT patient file" upload area (highlighted with a red box) with a folder icon. Below this are input fields for "Patient ID", "Date of birth" (format: mm/dd/yyyy), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown for "Reason for recording" (currently "Not selected").
- Notes section:** Includes a text area labeled "Notes on patient or recording".
- Device setup section:** Features dropdowns for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a heart icon and the text "um C3+".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections. The top section, 'Uploaded By', shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'. The middle section, 'Recording Summary', lists details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes 'View recording' and 'Download' buttons. The bottom section, 'Holter Reports', shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right of this section are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'. The bottom of the sidebar contains version information and a user profile for 'Cortrium Test'.

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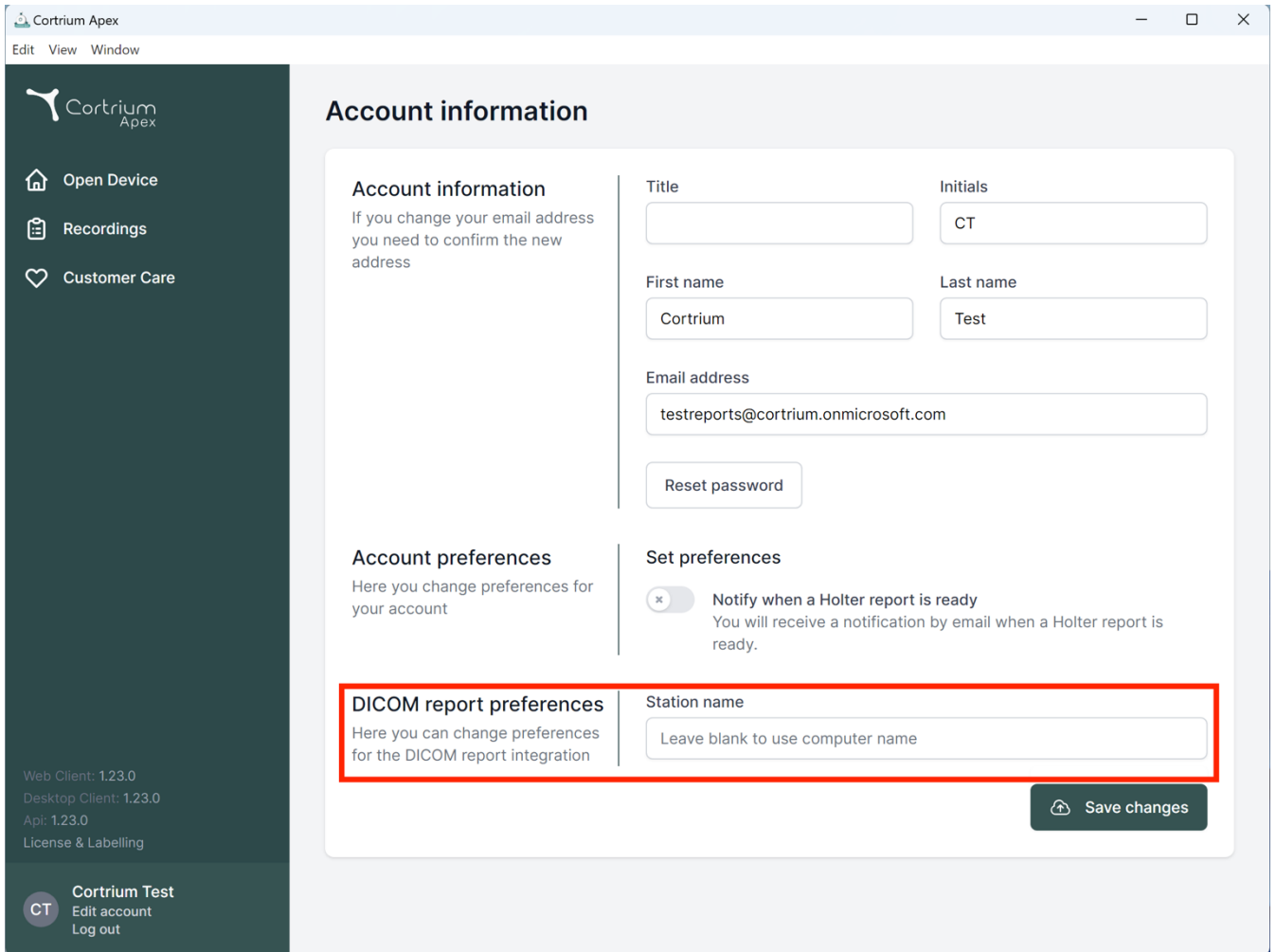Cortrium Apex
Edit View Window

Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

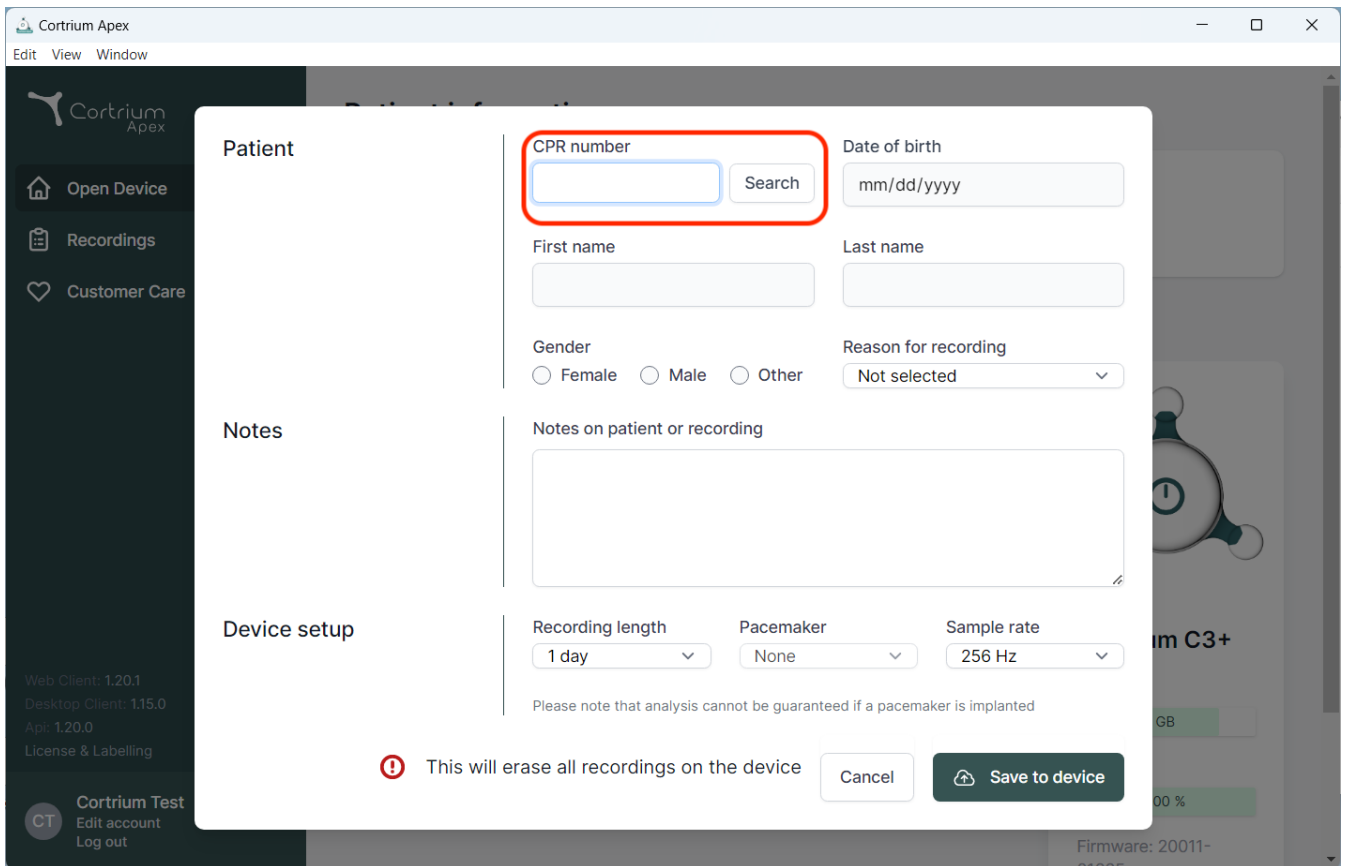
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a search form. The form includes the following fields and controls:

- CPR number:** A text input field with a blue border, highlighted by a red rectangle. To its right is a "Search" button.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup fields, a warning message states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom of the modal, there is a red warning icon followed by the text "This will erase all recordings on the device", a "Cancel" button, and a "Save to device" button.

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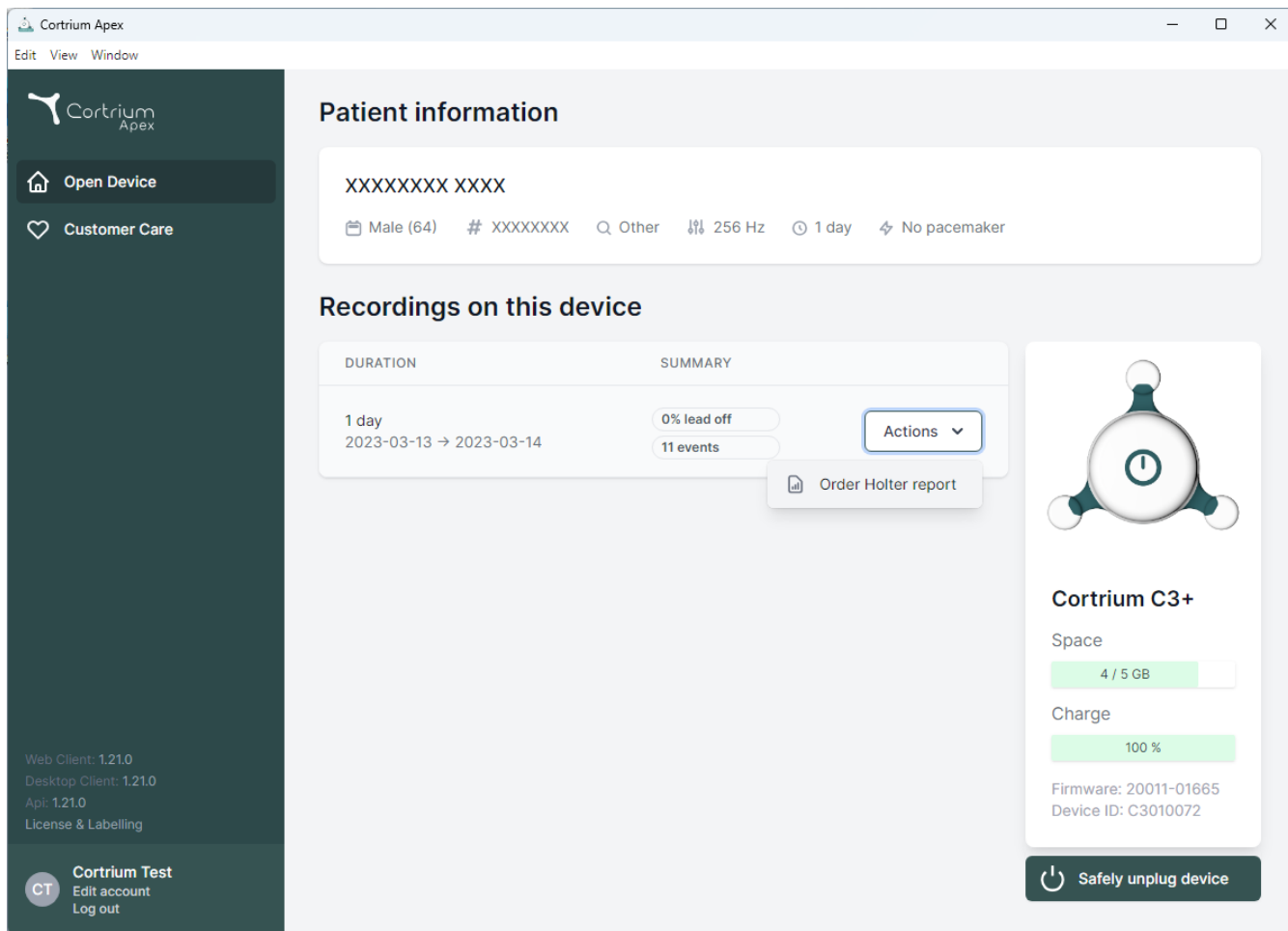The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', 'Customer Care', and version information (Web Client: 1.21.0, Desktop Client: 1.21.0, Api: 1.21.0, License & Labelling). The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions Order Holter report

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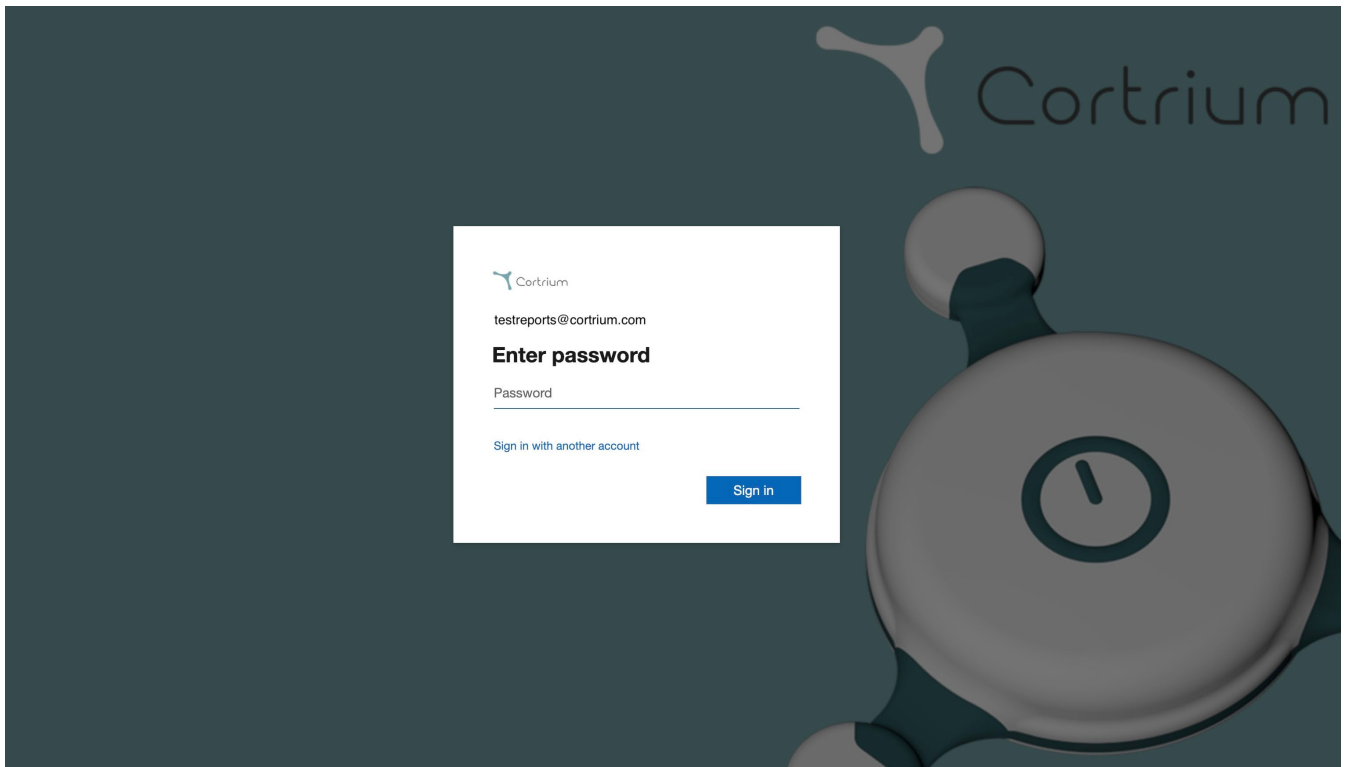
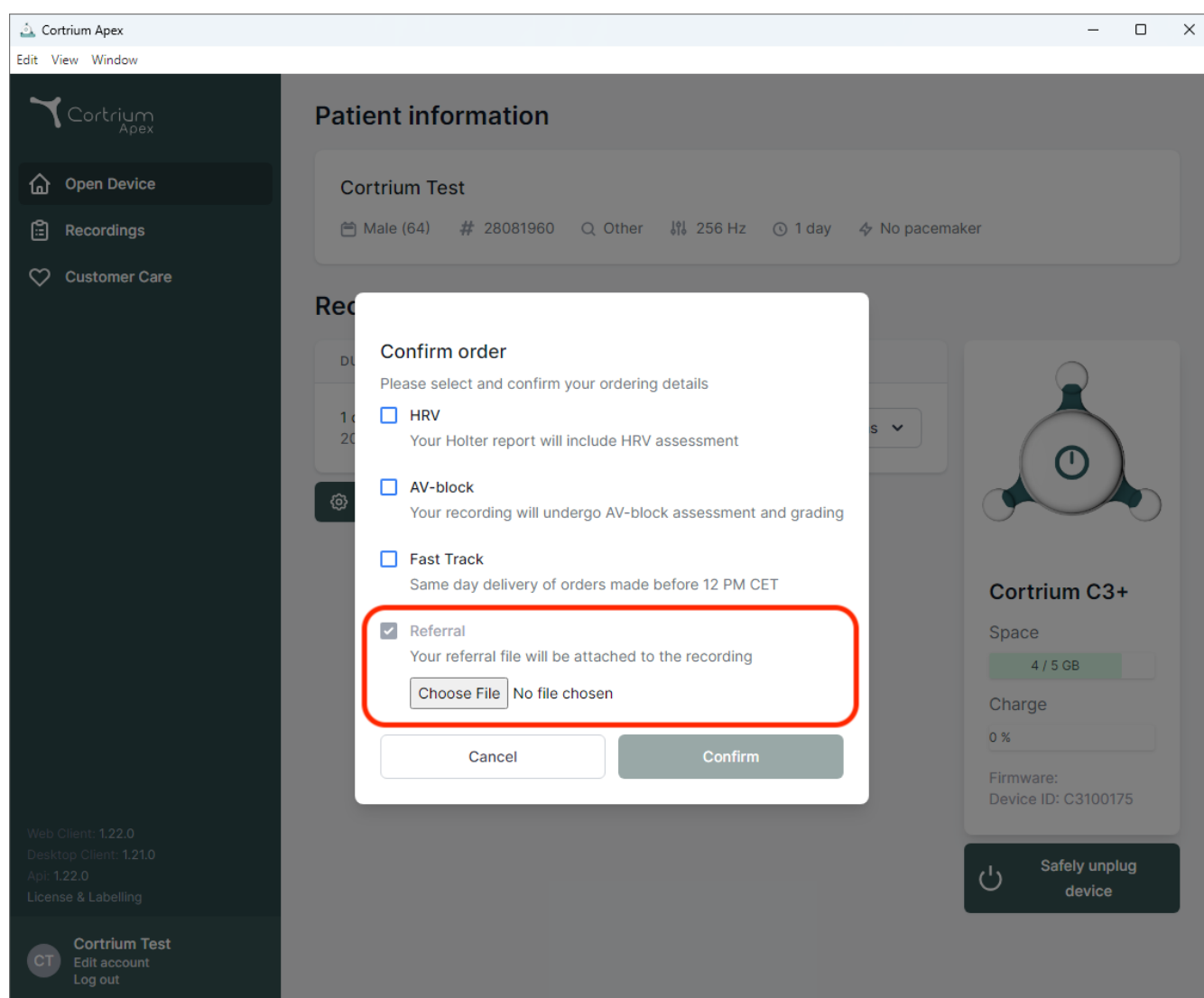
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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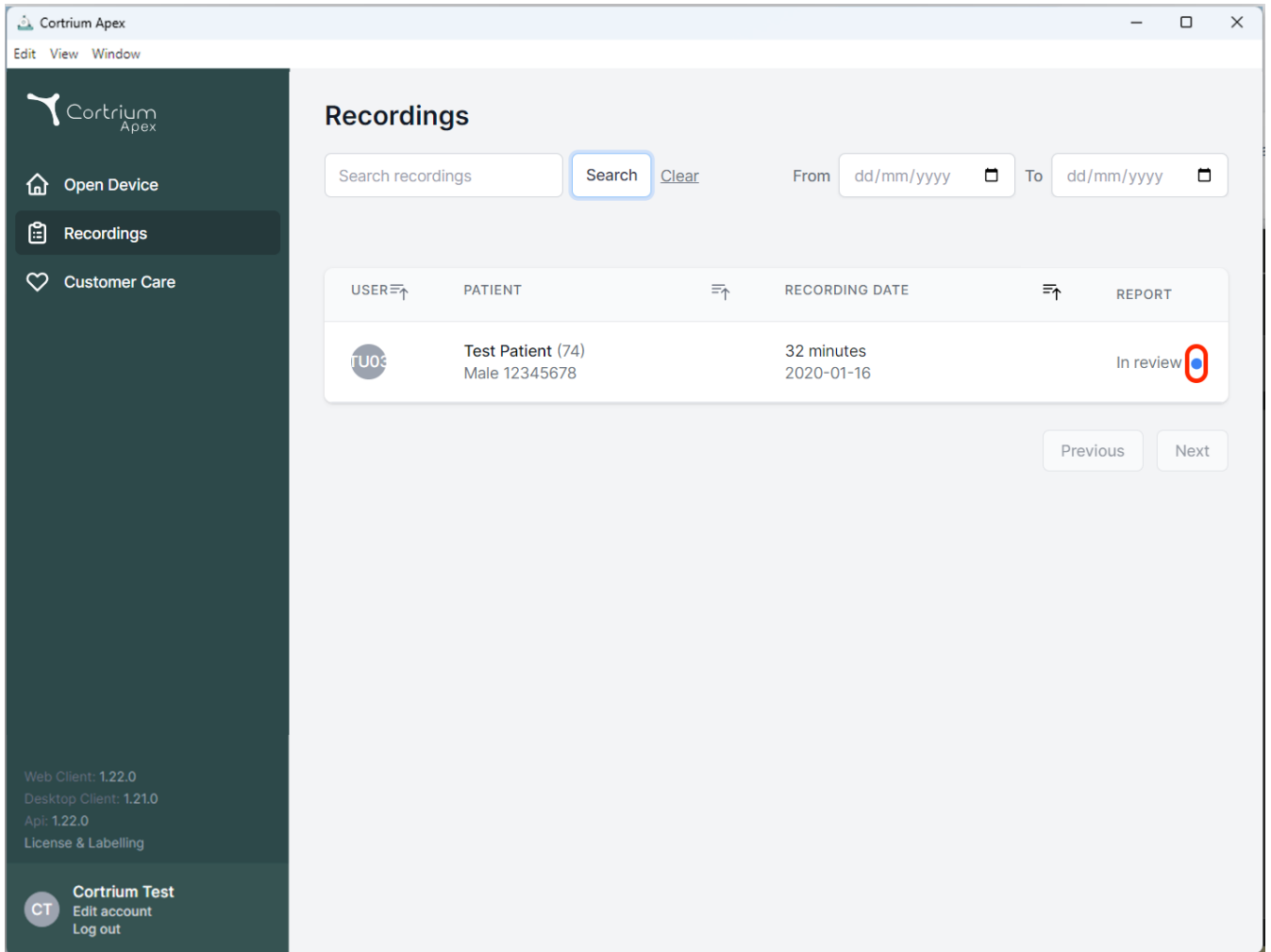
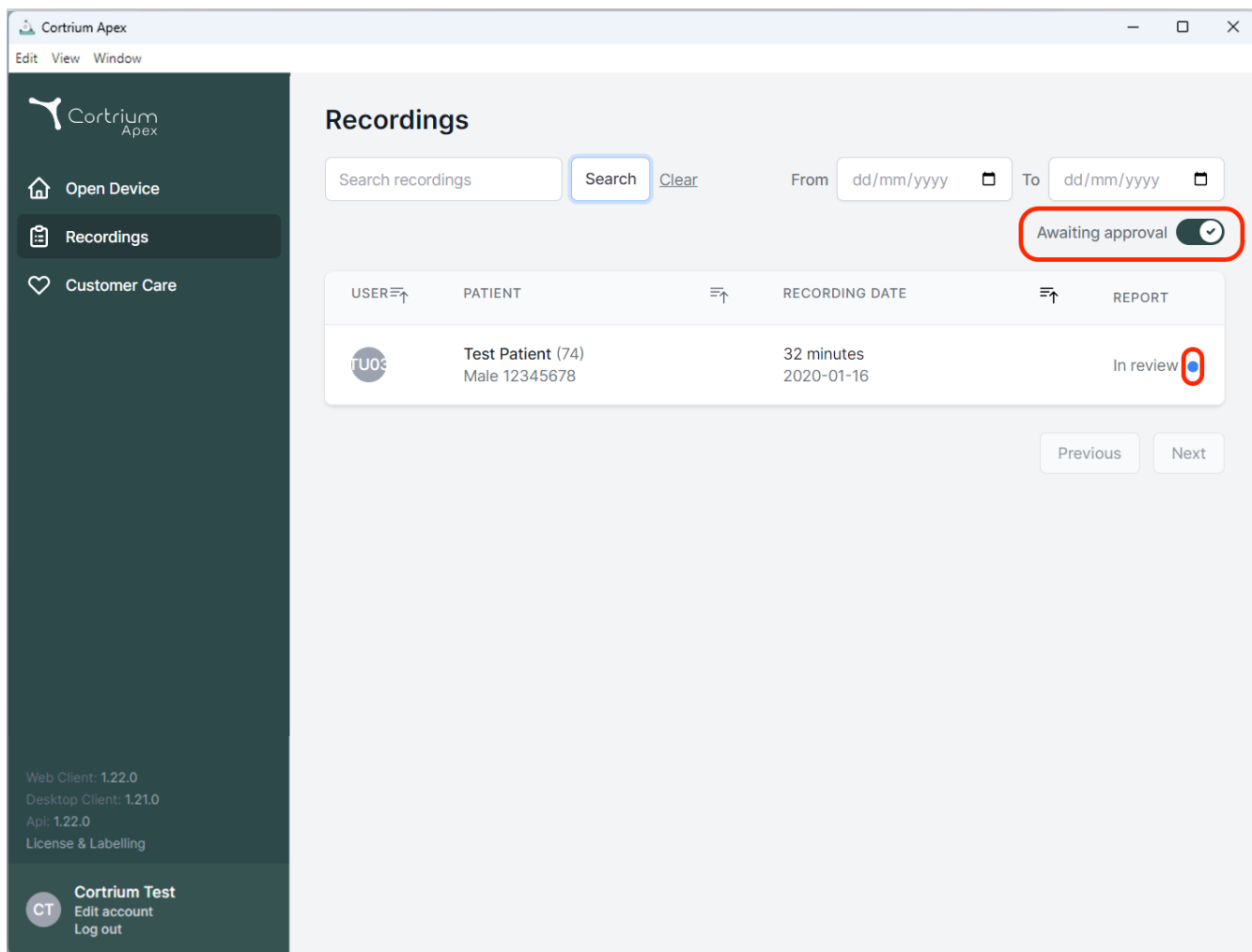
Edit View Window

Recordings

Search recordings Search Clear From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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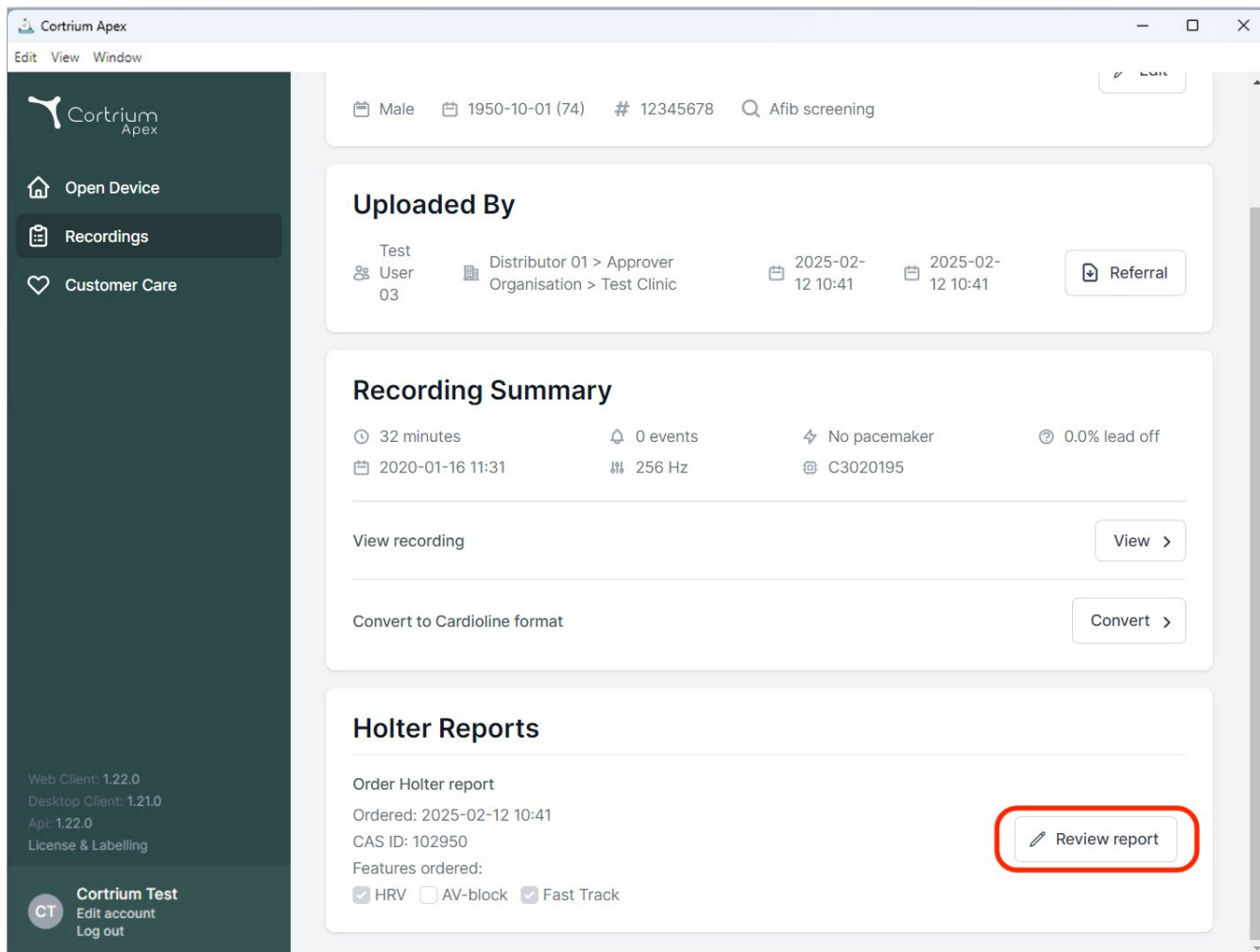The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the "Uploaded By" section, indicating the test was performed by User 03 on 2025-02-12 10:41. The "Recording Summary" section provides details: 32 minutes duration, 0 events, 256 Hz, and 0.0% lead off. It includes buttons for "View recording" and "Convert to Cardioline format". The "Holter Reports" section shows the report was ordered on 2025-02-12 10:41 with CAS ID 102950. A red box highlights the "Review report" button. The bottom left corner shows version information and user options.

Cortrium Apex

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

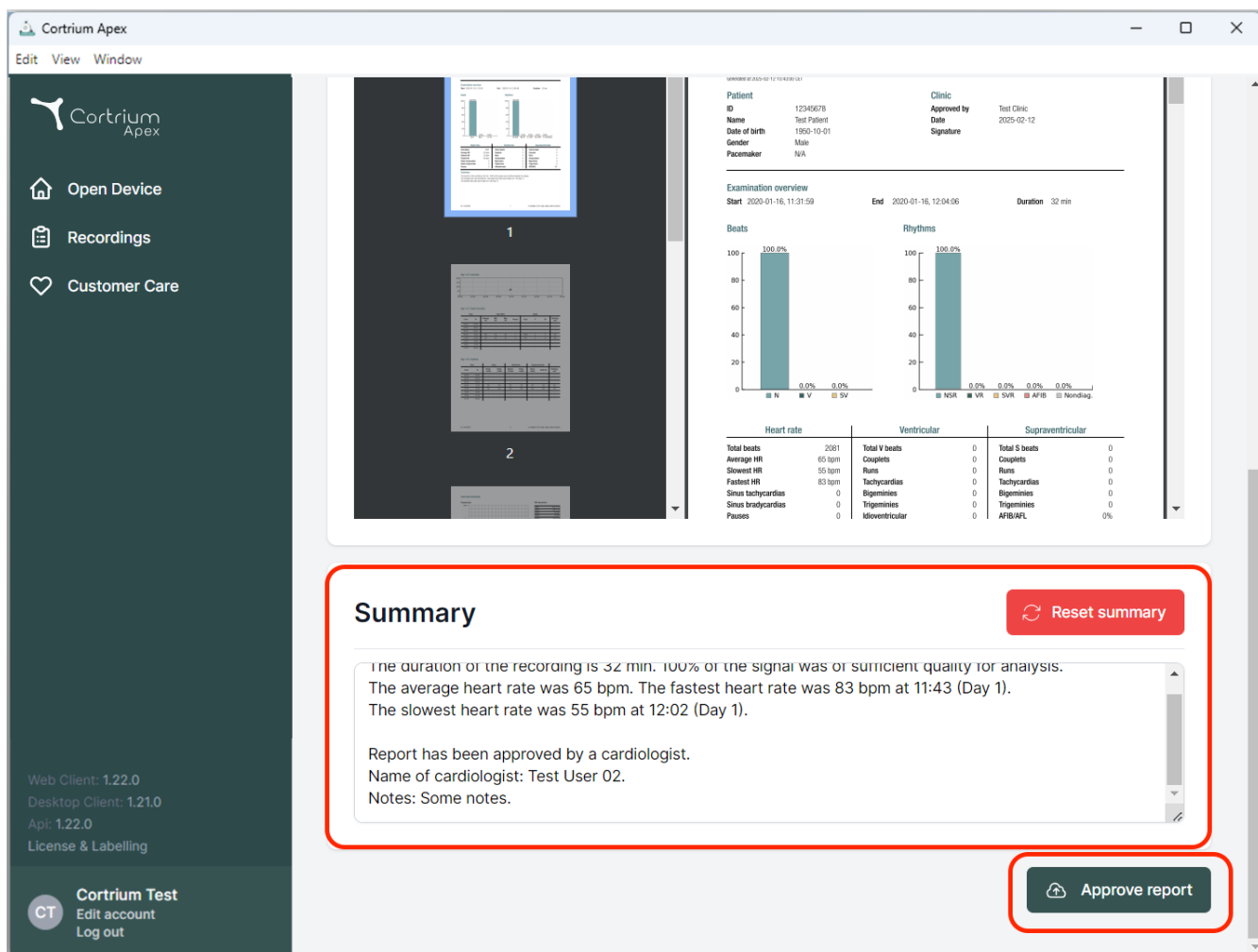
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Apl: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

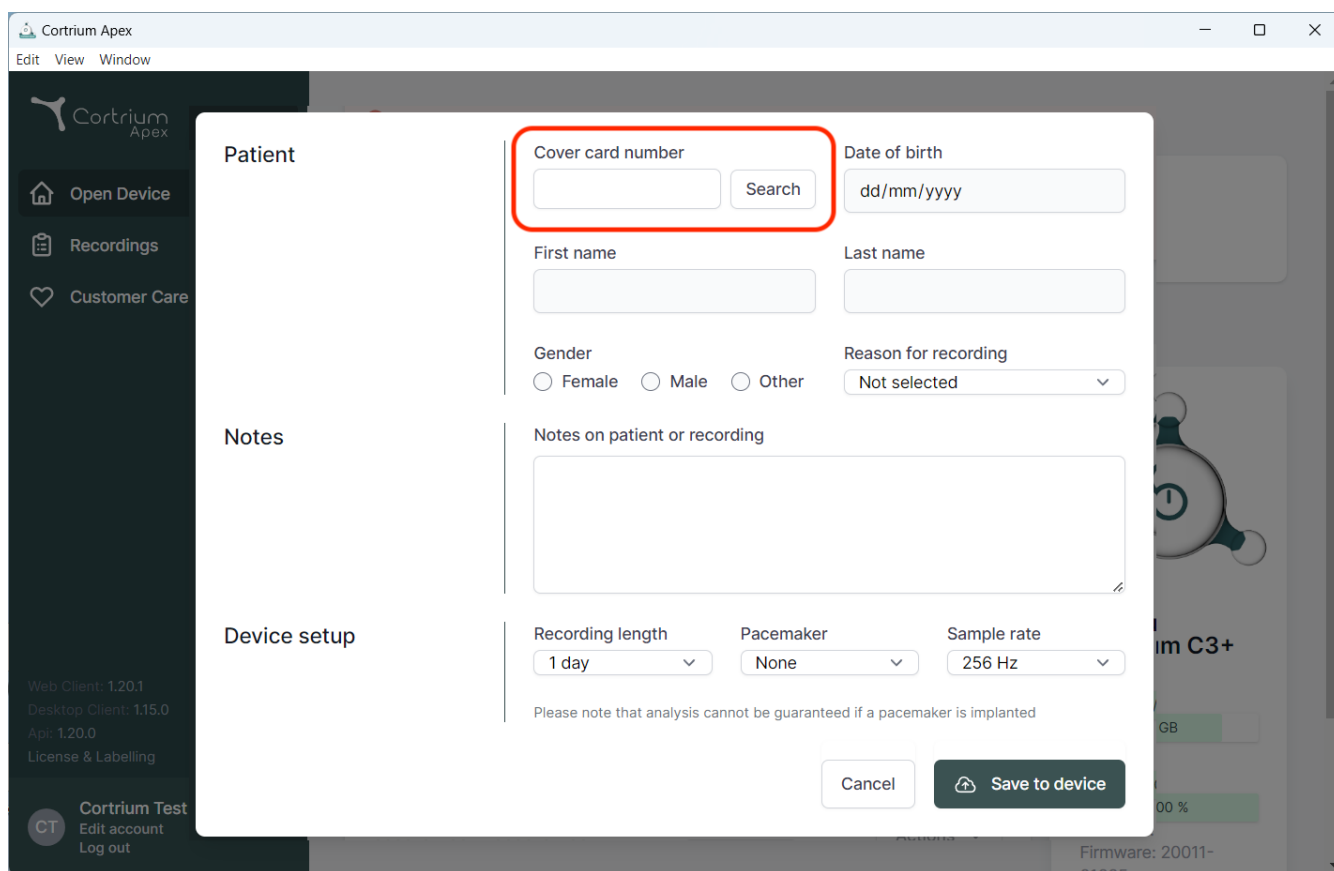
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, allowing for data entry. The form is organized into sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test", "Edit account", "Log out", and "Firmware: 20011-0100".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dánia
E-mail: info@cortrium.com • Weboldal: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47