Cortrium Apex 1.26

„Cortrium Apex 1.26“

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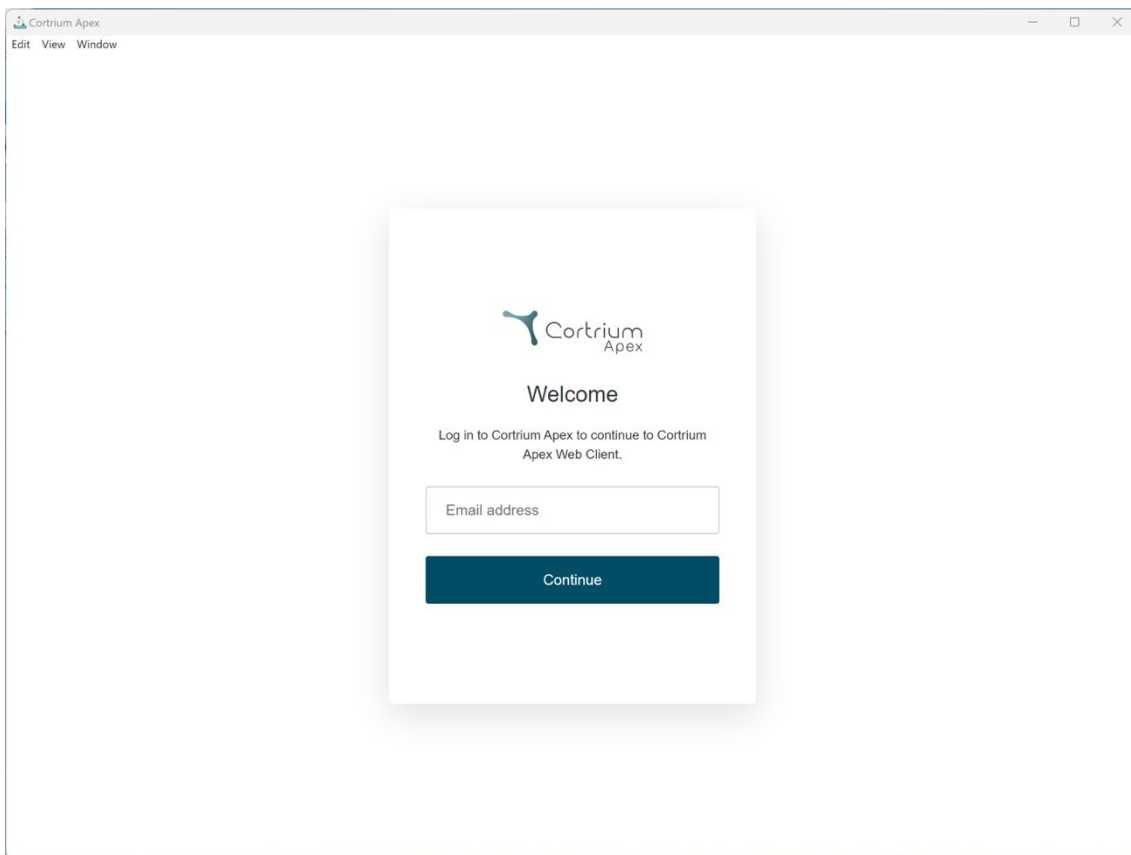
 Cortrium
Apex

Welcome

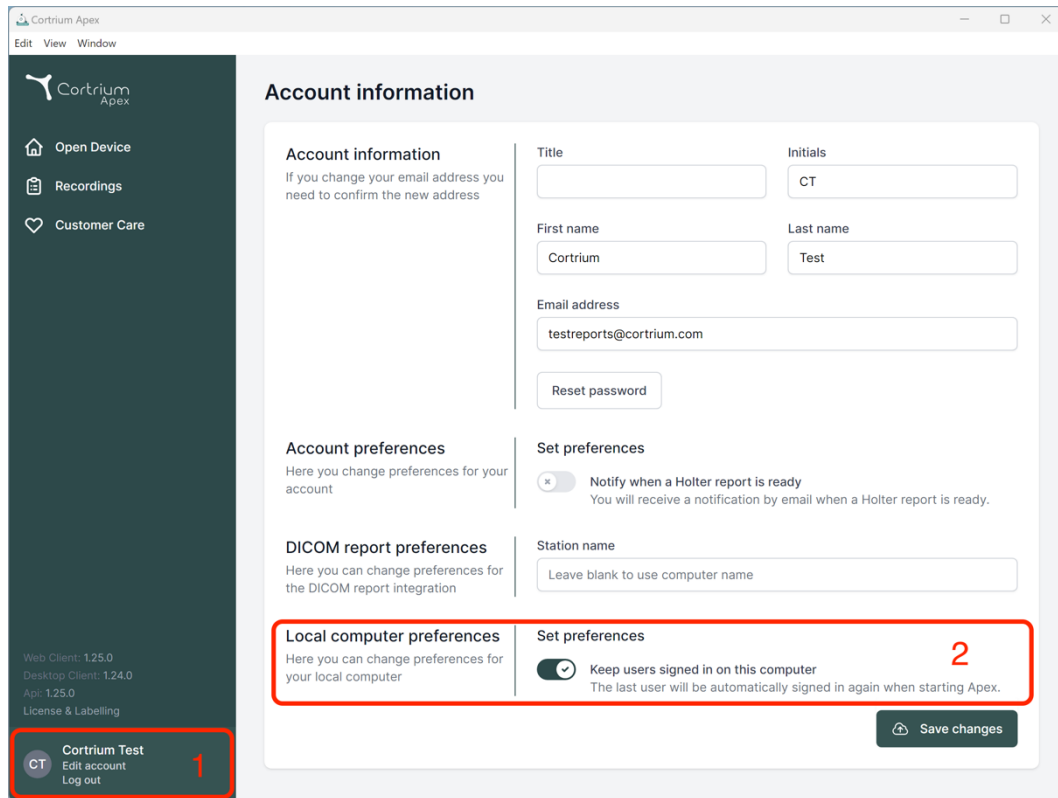
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section includes a 'Set preferences' area with a toggle for 'Keep users signed in on this computer', which is currently turned on and marked with a red '2'. A 'Save changes' button is located at the bottom right of the preferences section.

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Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

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Station name

Local computer preferences

Here you can change preferences for your local computer

Set preferences

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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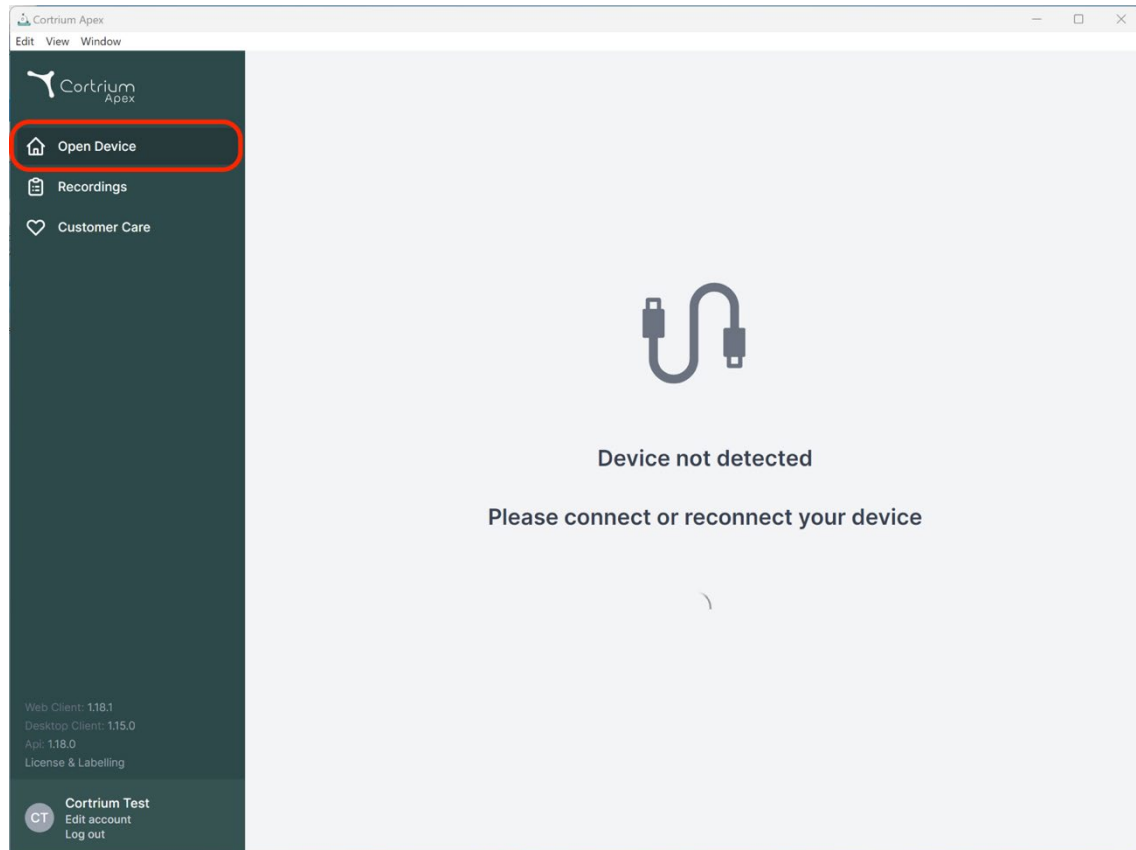
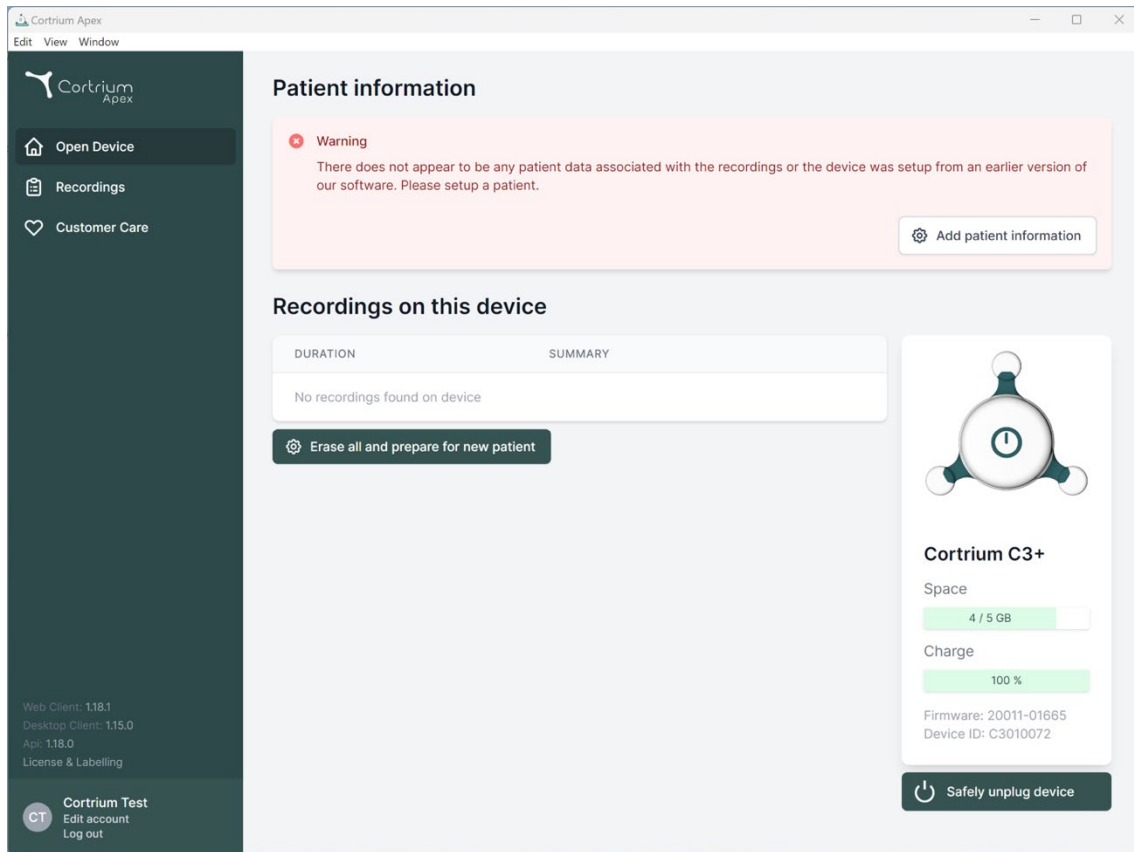
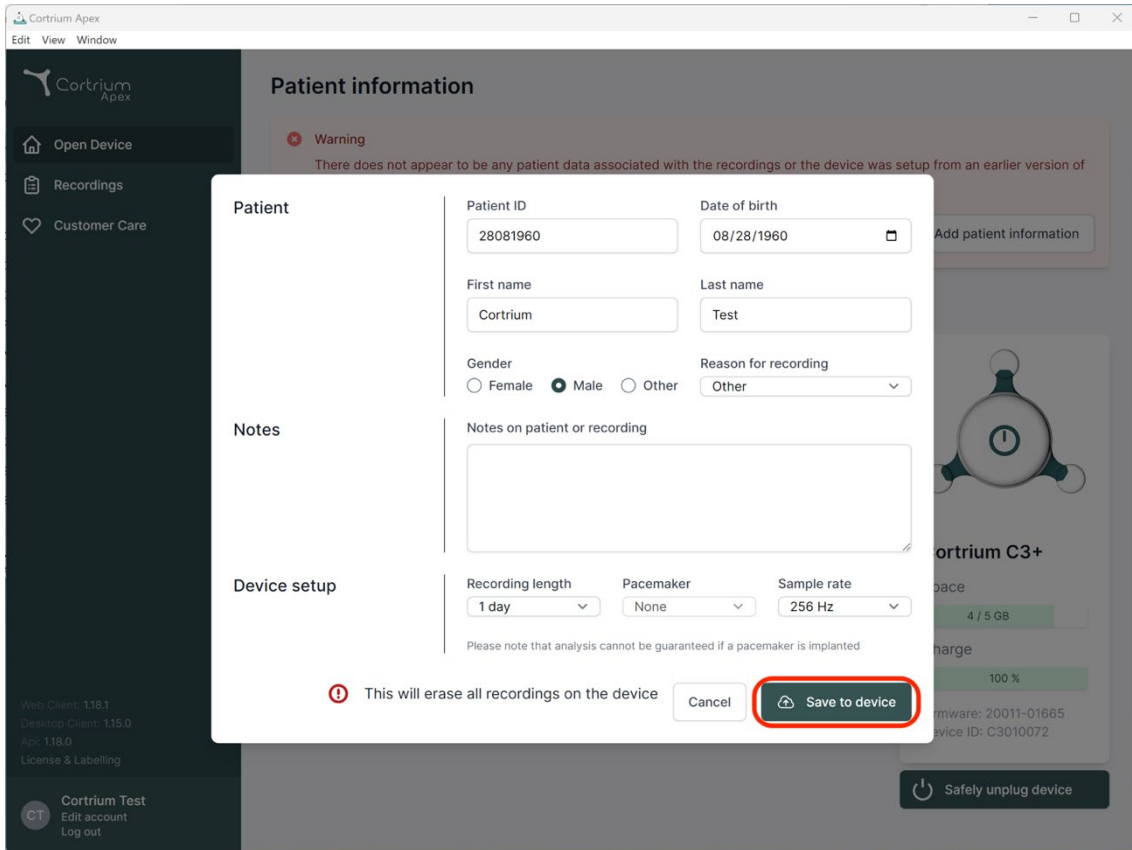
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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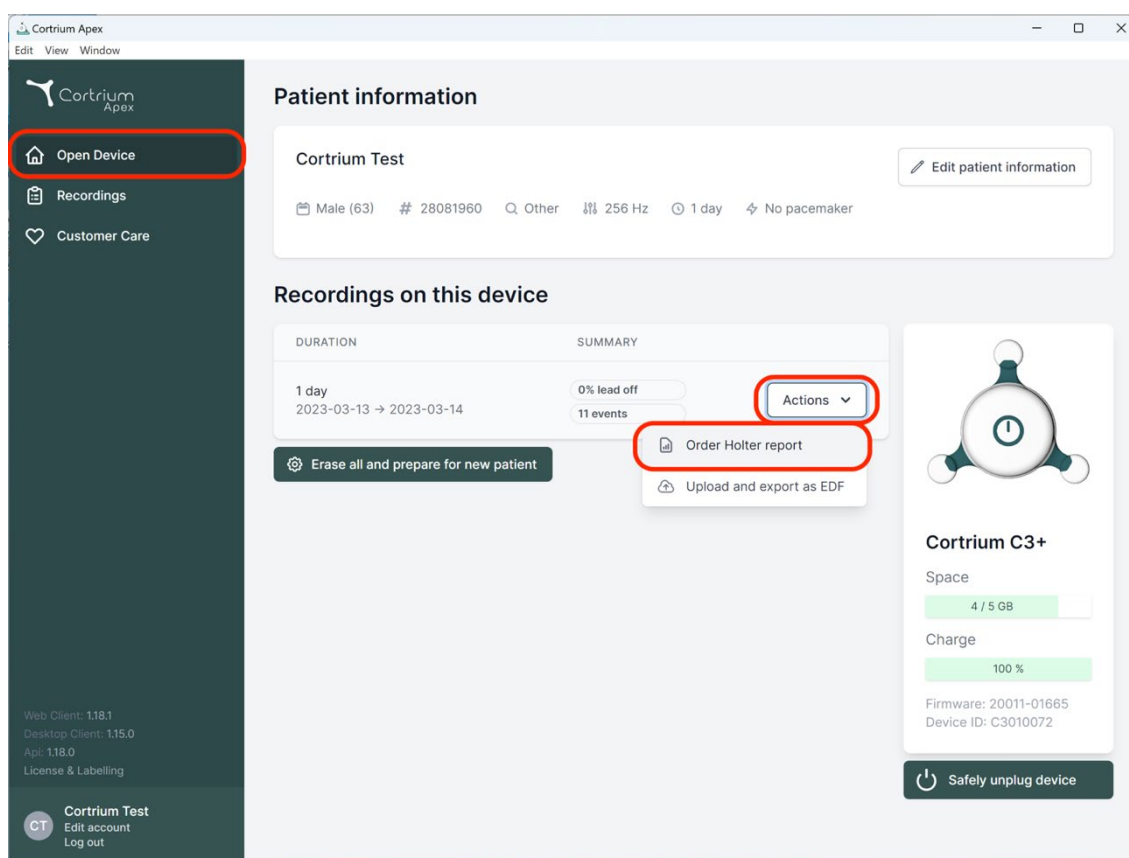
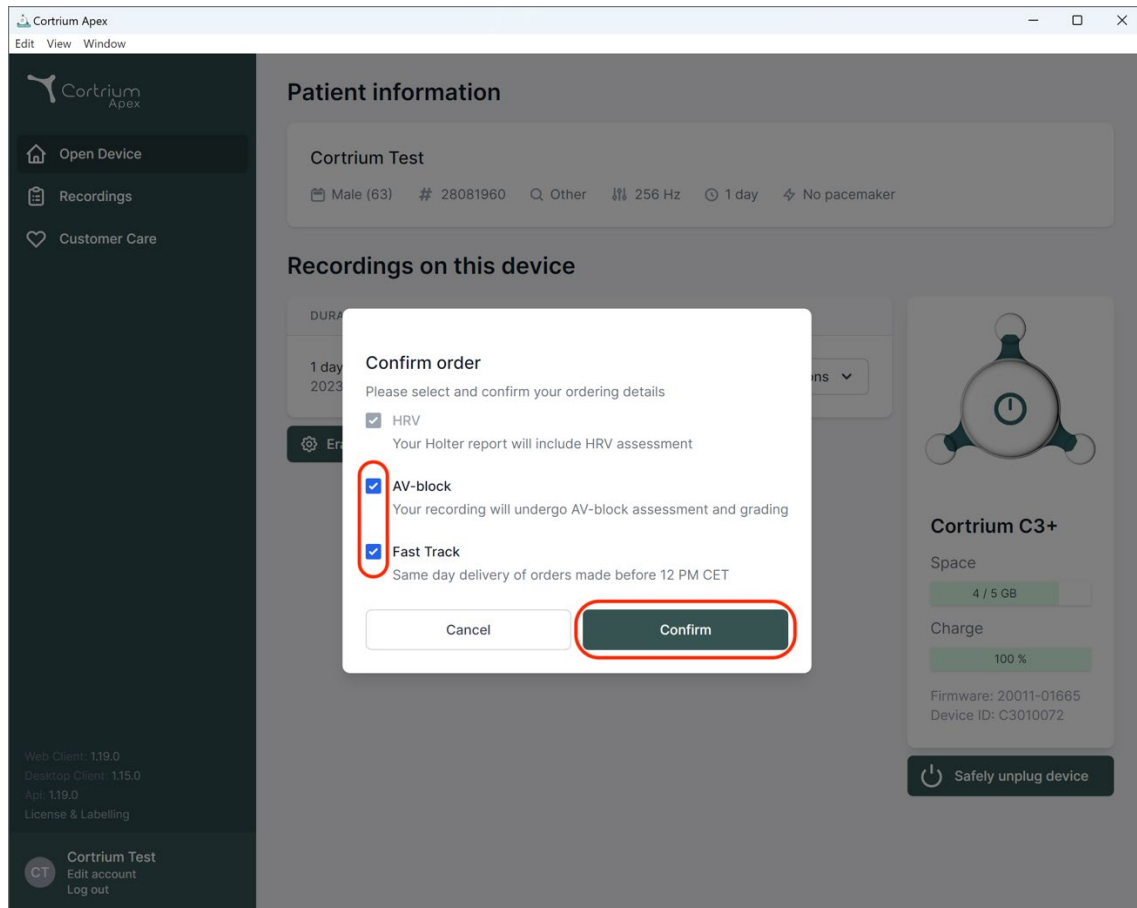
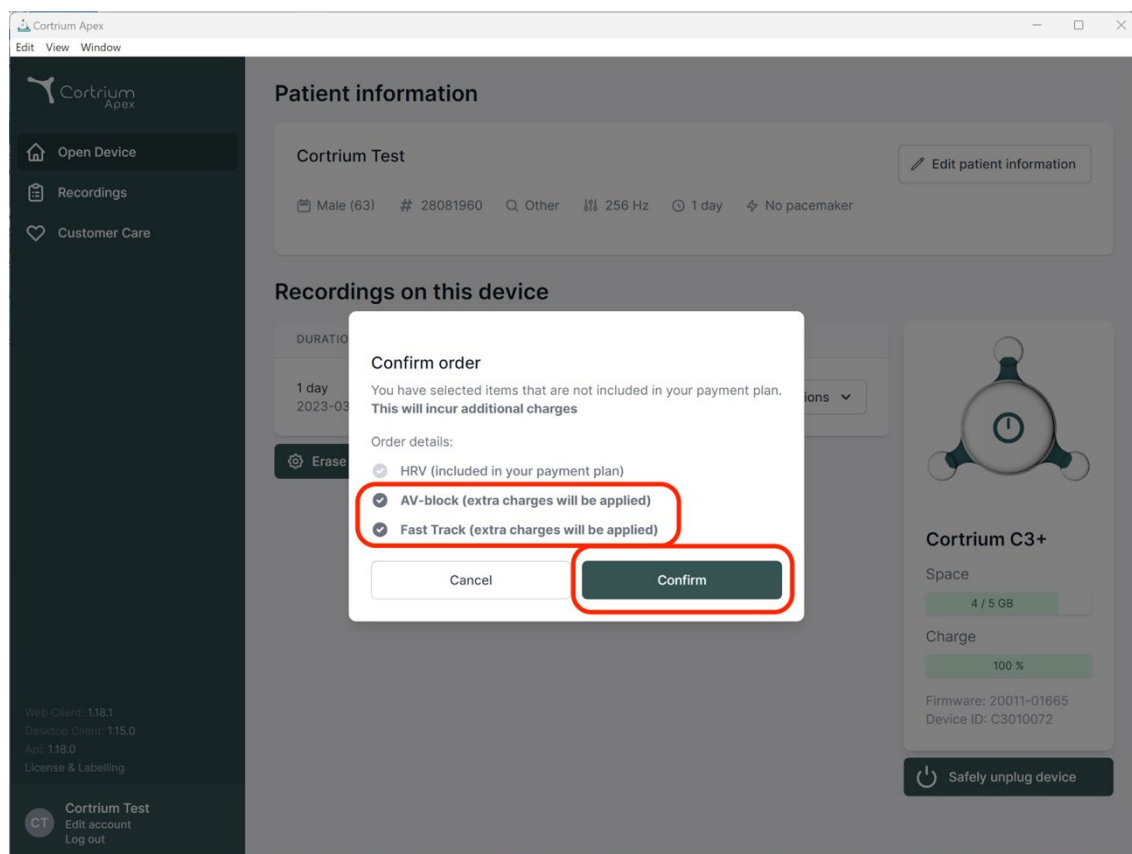
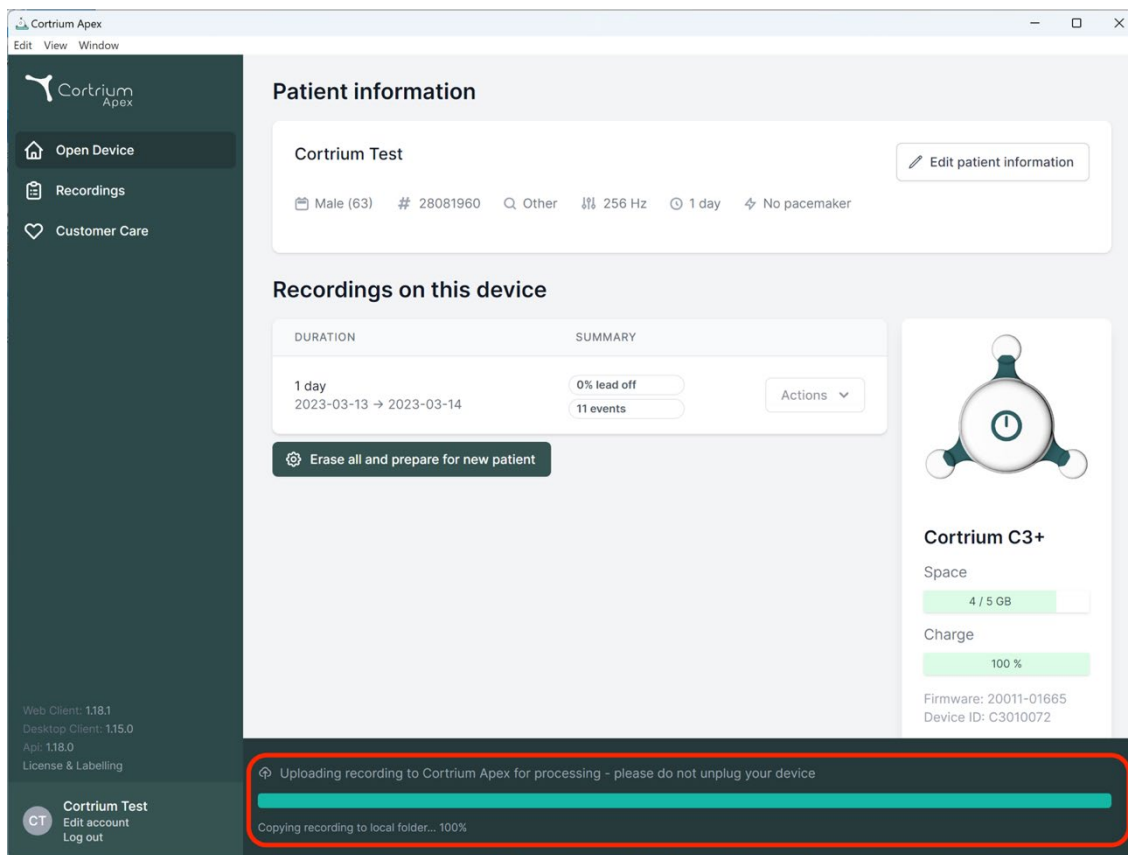
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Patient information' and displays data for 'Cortrium Test', including 'Male (63)', ID '28081960', 'Other' category, '256 Hz' sampling rate, '1 day' duration, and 'No pacemaker'. An 'Edit patient information' button is present. Below this is the 'Recordings on this device' section, showing a table with columns 'DURATION' and 'SUMMARY'. A recording for '1 day' (2023-03-13 to 2023-03-14) is listed with '0% lead off' and '11 events'. An 'Actions' dropdown menu is next to it. A button 'Erase all and prepare for new patient' is below the table. To the right of the table is a device status card for 'Cortrium C3+' showing 'Space: 4 / 5 GB', 'Charge: 100 %', 'Firmware: 20011-01665', and 'Device ID: C3010072'. At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' with a progress bar and 'Copying recording to local folder... 100%'.

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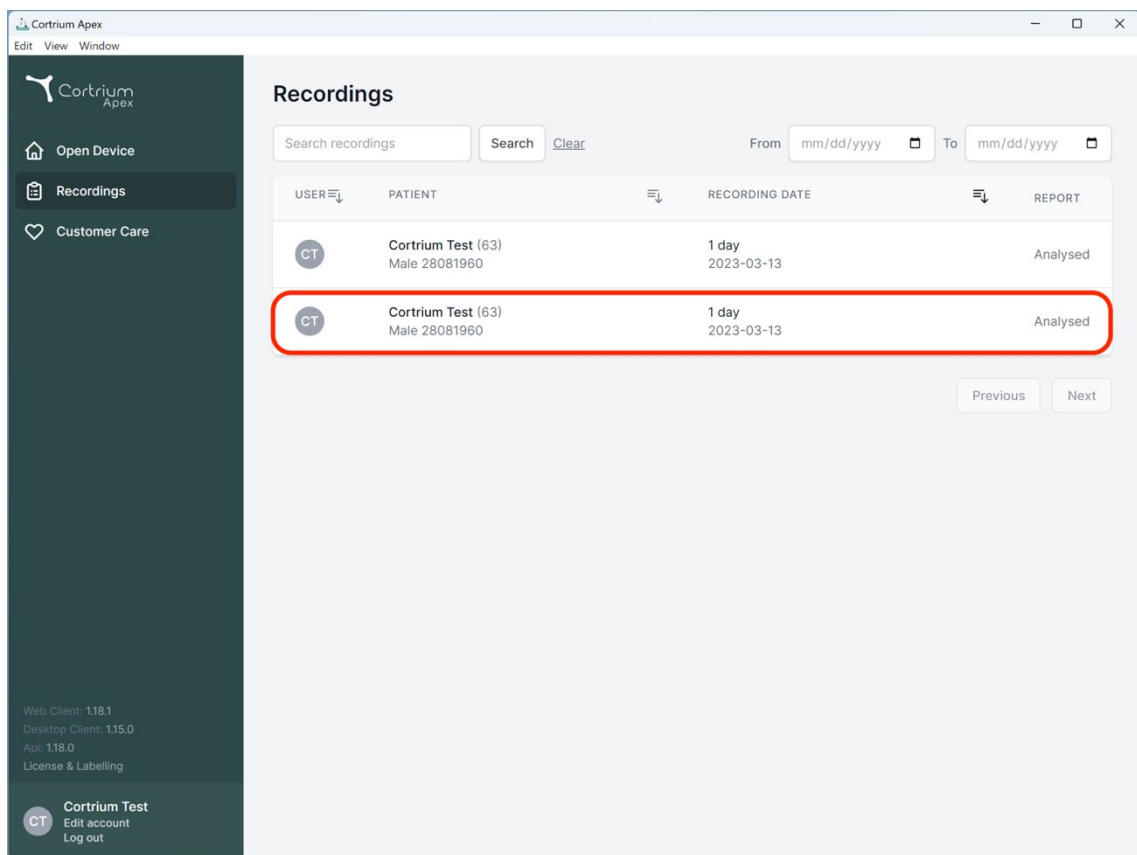
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area shows a table of recordings. The second row is highlighted with a red border.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

At the bottom of the sidebar, there is a footer with version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'App: 1.18.0', and 'License & Labelling'. At the very bottom of the sidebar, there is a 'Cortrium Test' section with 'Edit account' and 'Log out' links.

Cortrium Apex

Edit View Window

[Open Device](#)

[Recordings](#)

[Customer Care](#)

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT

Cortrium Test

Edit account
Log out

Recording details

Delete recording

Cortrium Test

Male

1960-08-28 (63)

28081960

Other

Edit

Uploaded By

Cortrium Test

Cortrium Internal > Cortrium

2024-01-30 12:31

Recording Summary

1 day

11 events

No pacemaker

0.0% lead off

2023-03-13 11:54

256 Hz

C3100175

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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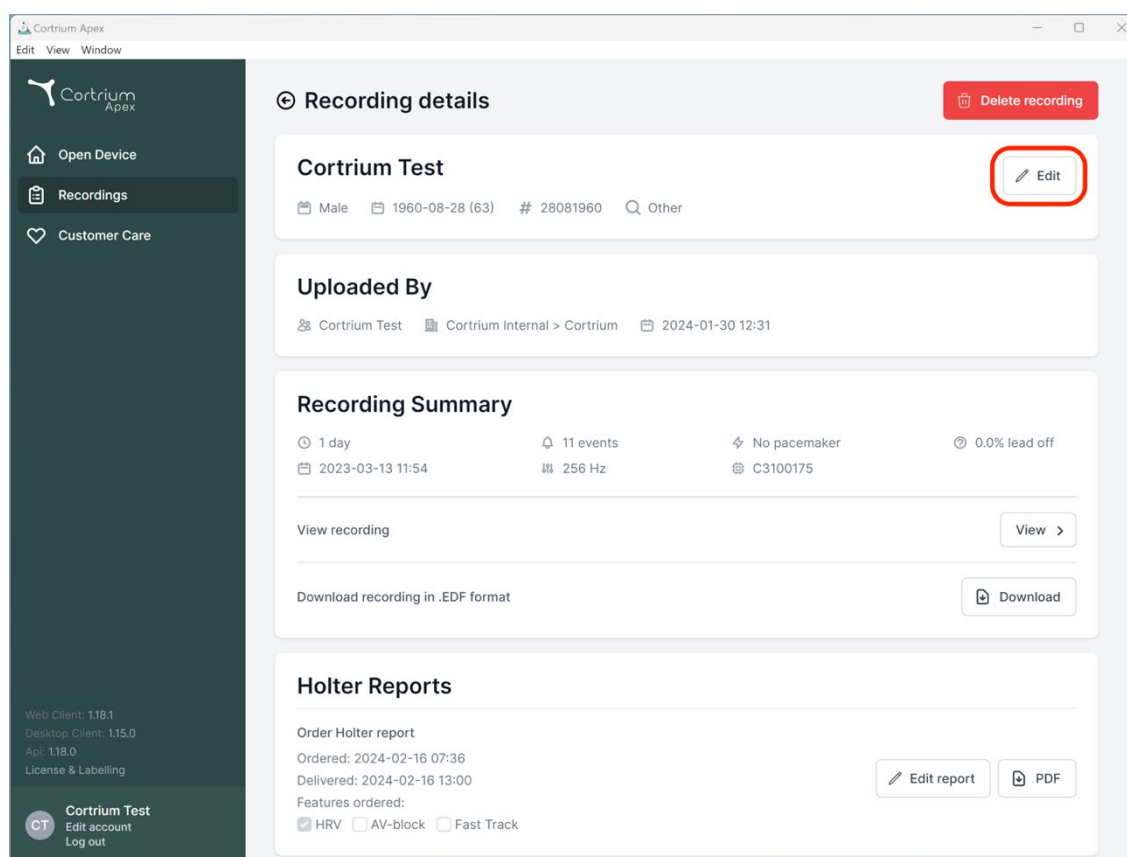
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

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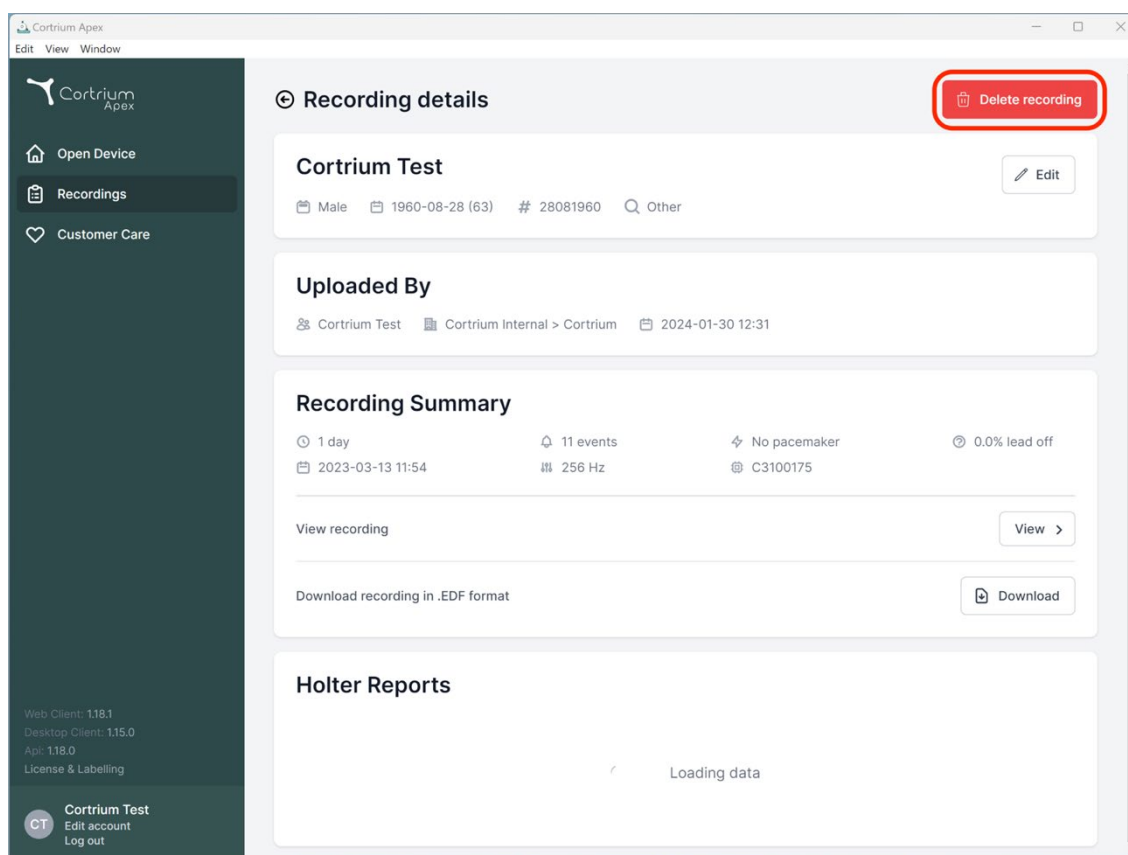
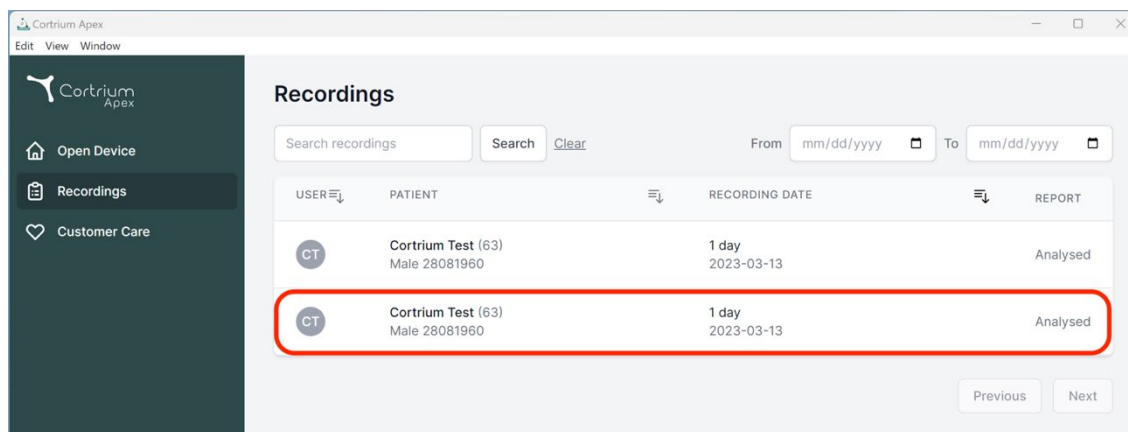
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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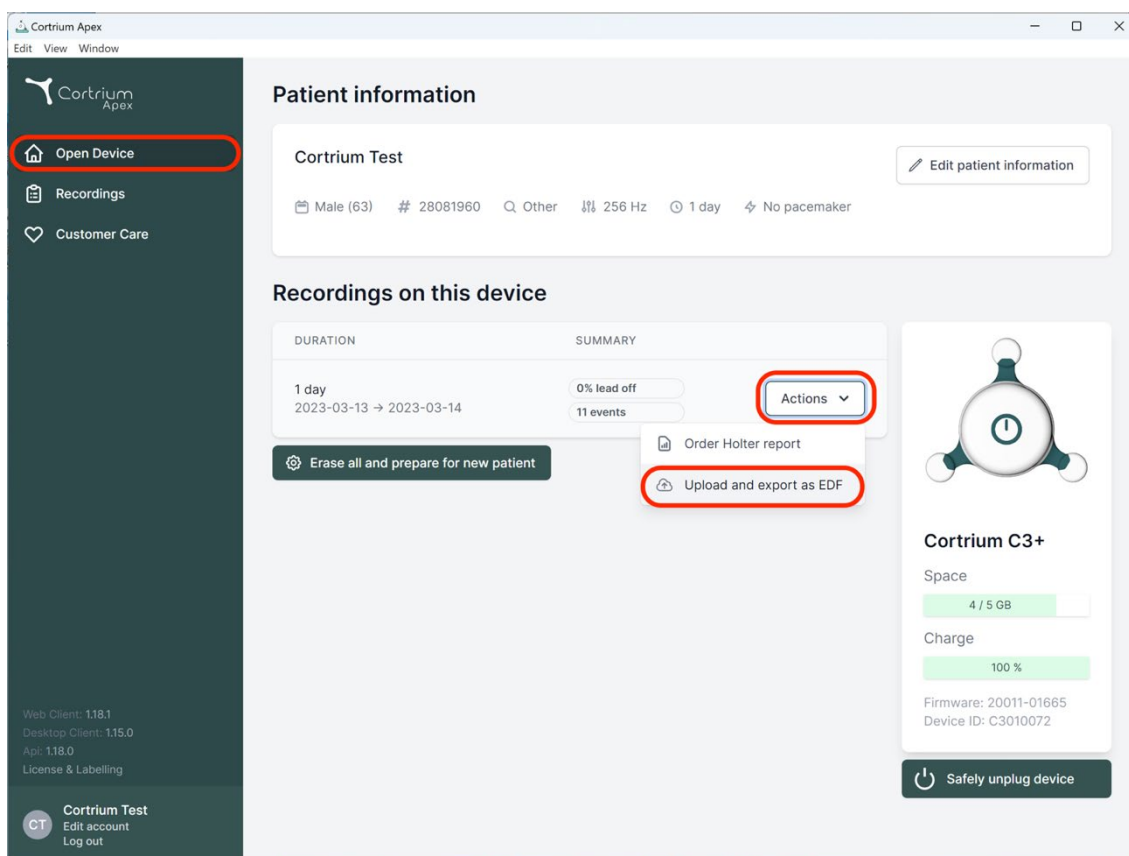
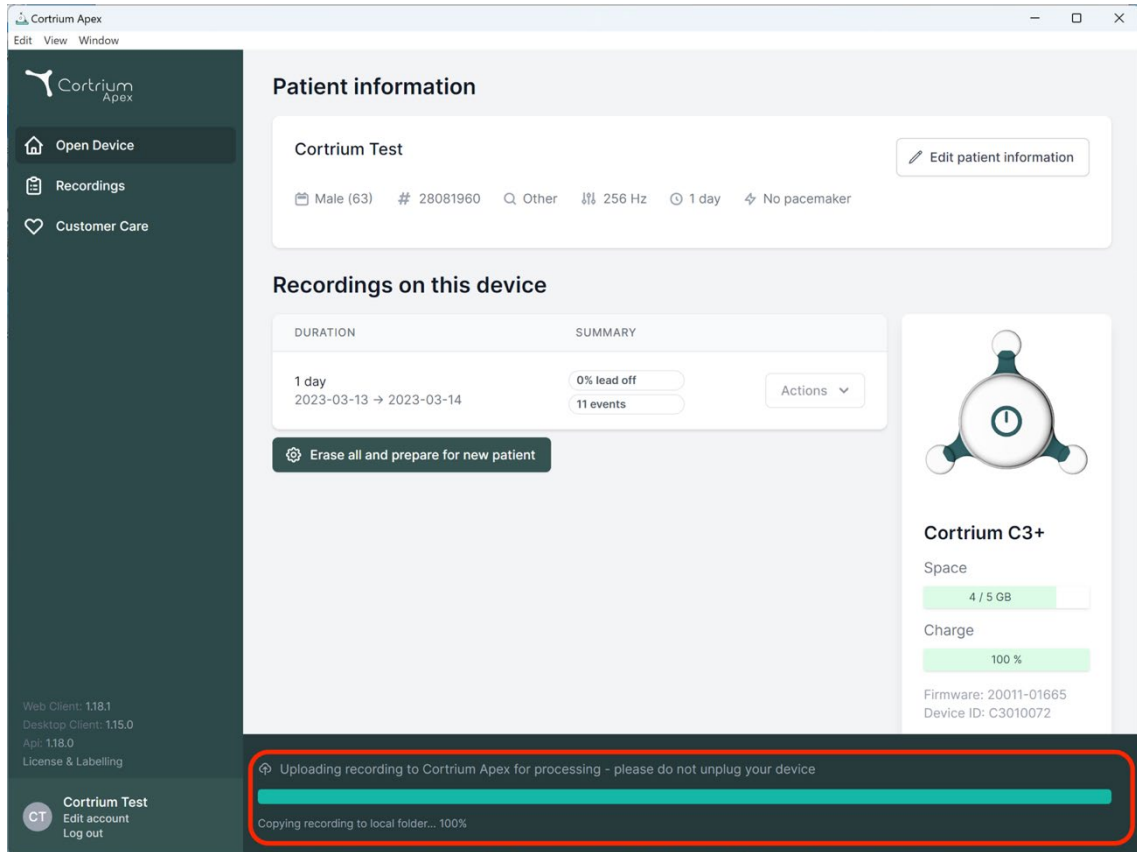
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

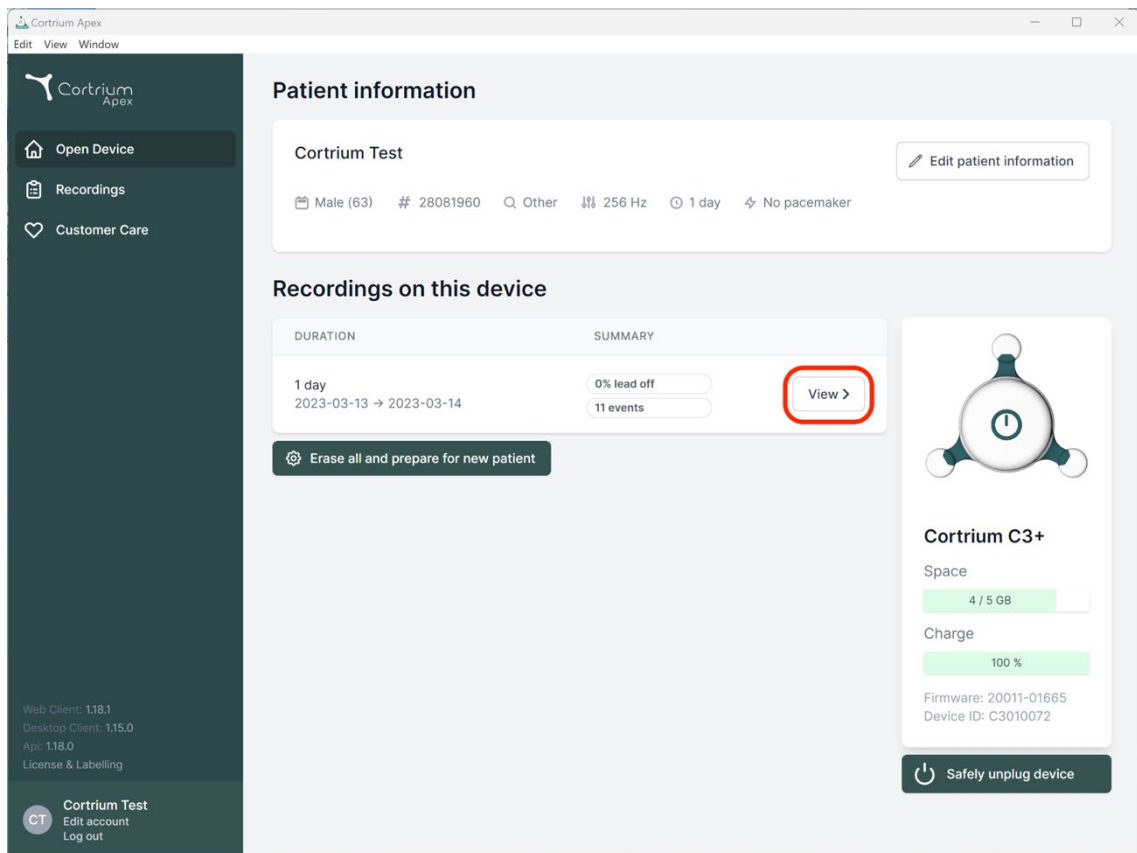
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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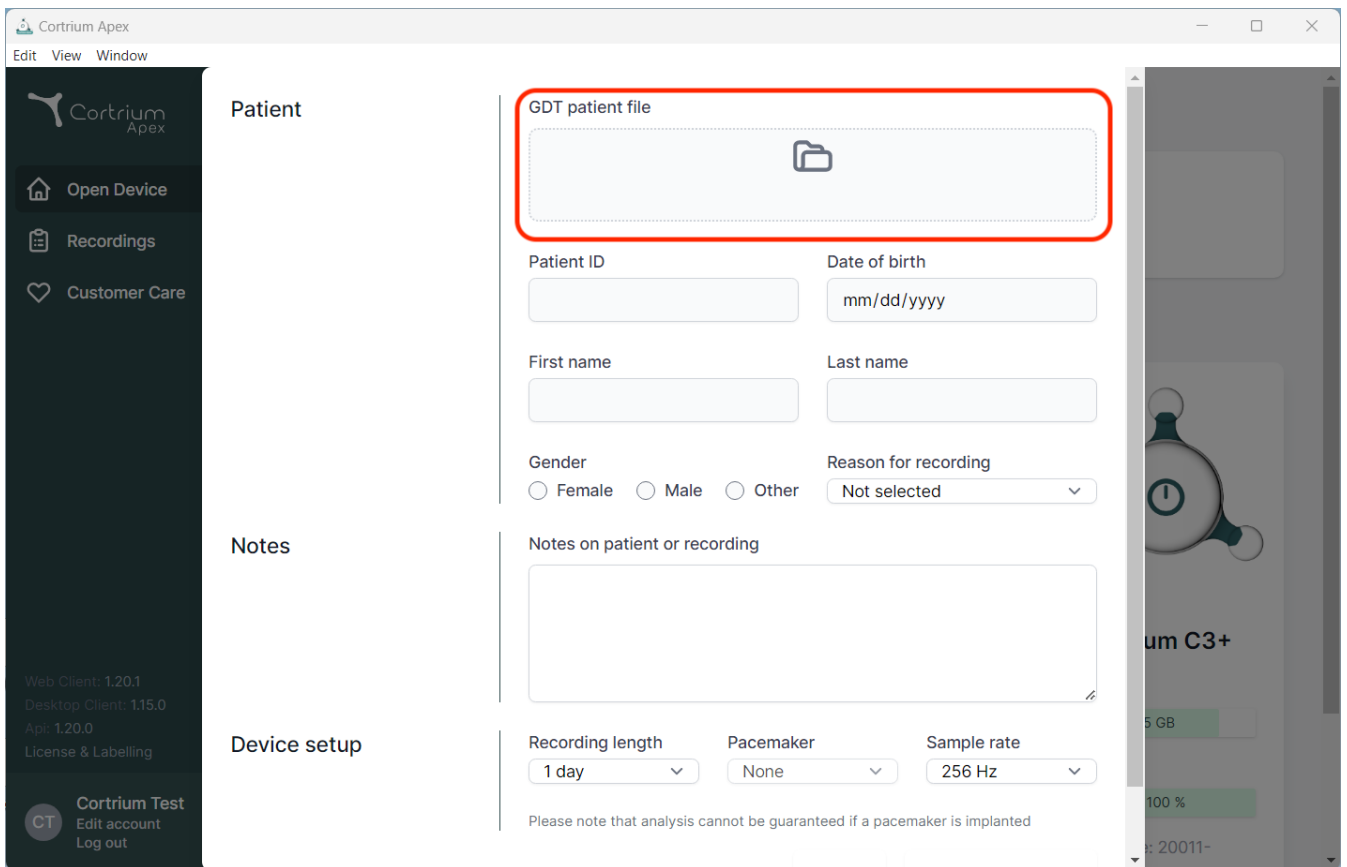
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

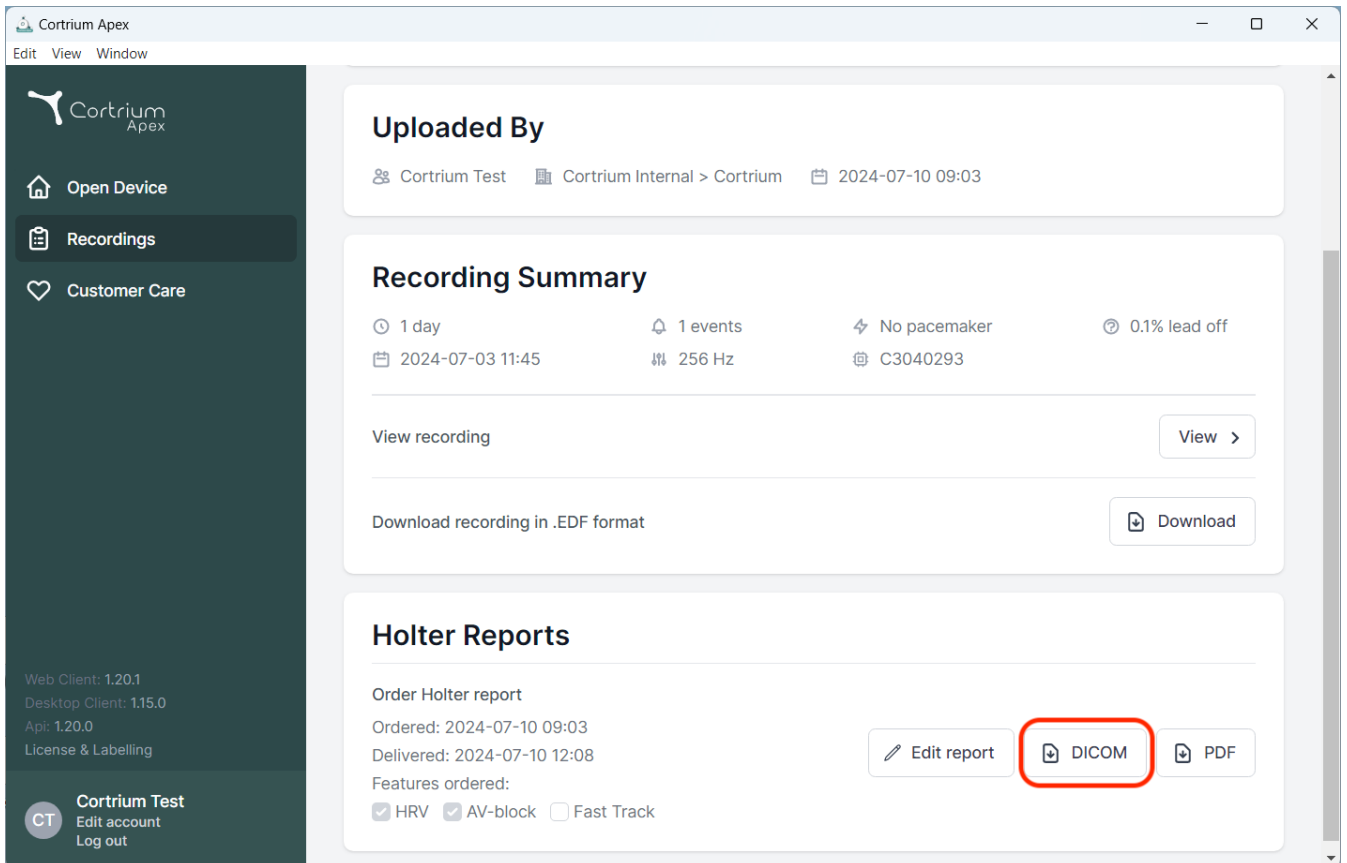
The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

- Patient section:** Contains a "GDT patient file" upload area (highlighted with a red box) with a folder icon. Below it are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently "Not selected").
- Notes section:** Contains a text area labeled "Notes on patient or recording".
- Device setup section:** Contains dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). Below these is a disclaimer: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a heart icon and the text "um C3+", "5 GB", "100 %", and "20011-".

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

Uploaded By
Cortrium Test | Cortrium Internal > Cortrium | 2024-07-10 09:03

Recording Summary
🕒 1 day | 📅 2024-07-03 11:45 | 🔔 1 events | 📡 256 Hz | ⚡ No pacemaker | 📶 0.1% lead off | 🏷️ C3040293

View recording [View >]

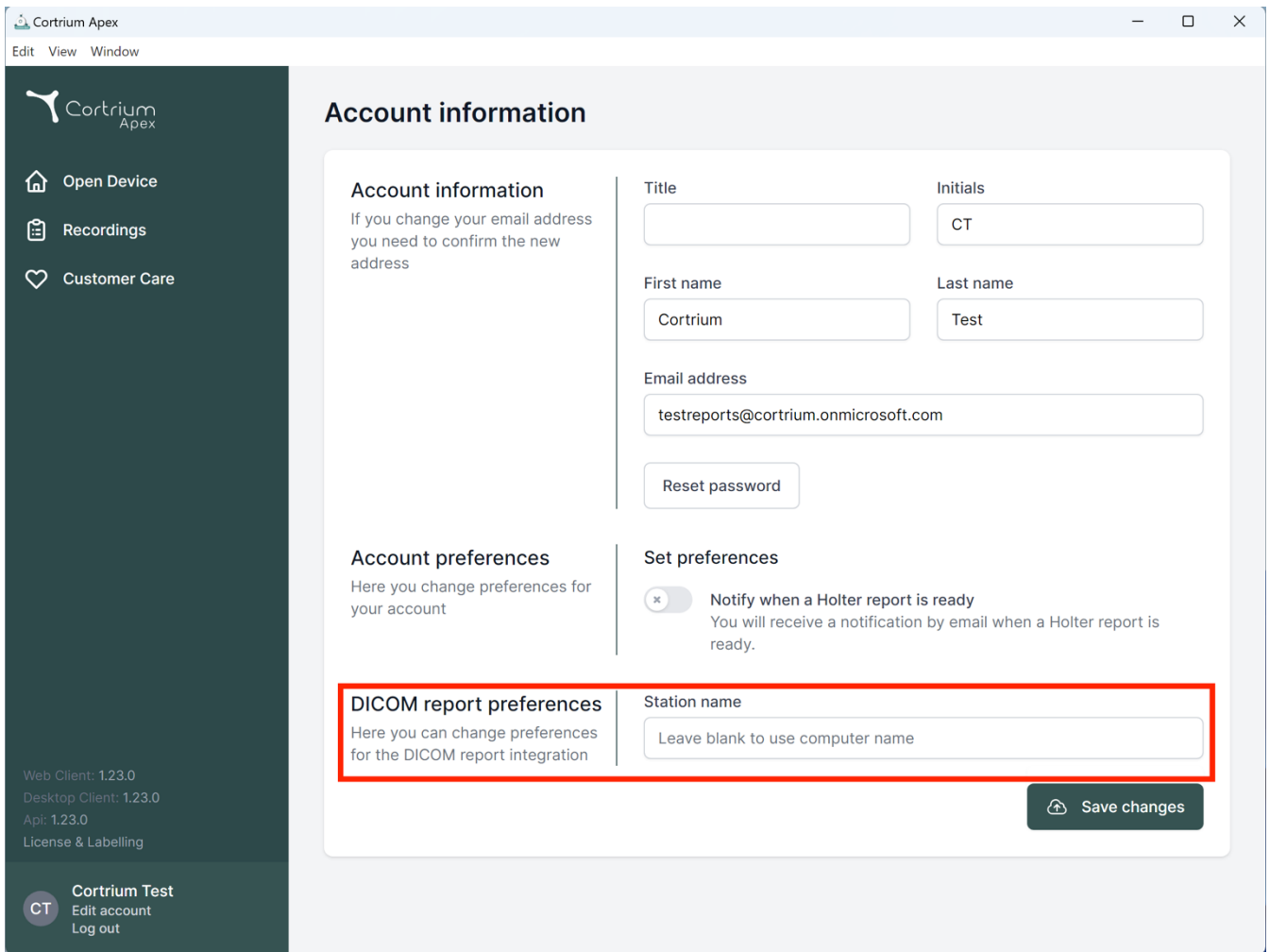
Download recording in .EDF format [Download]

Holter Reports
Order Holter report
Ordered: 2024-07-10 09:03
Delivered: 2024-07-10 12:08
Features ordered:
☒ HRV ☒ AV-block ☐ Fast Track

Buttons: [Edit report] [DICOM] [PDF]. The 'DICOM' button is highlighted with a red rectangle.

Footer (left sidebar):
Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling
CT Cortrium Test
Edit account
Log out

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Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

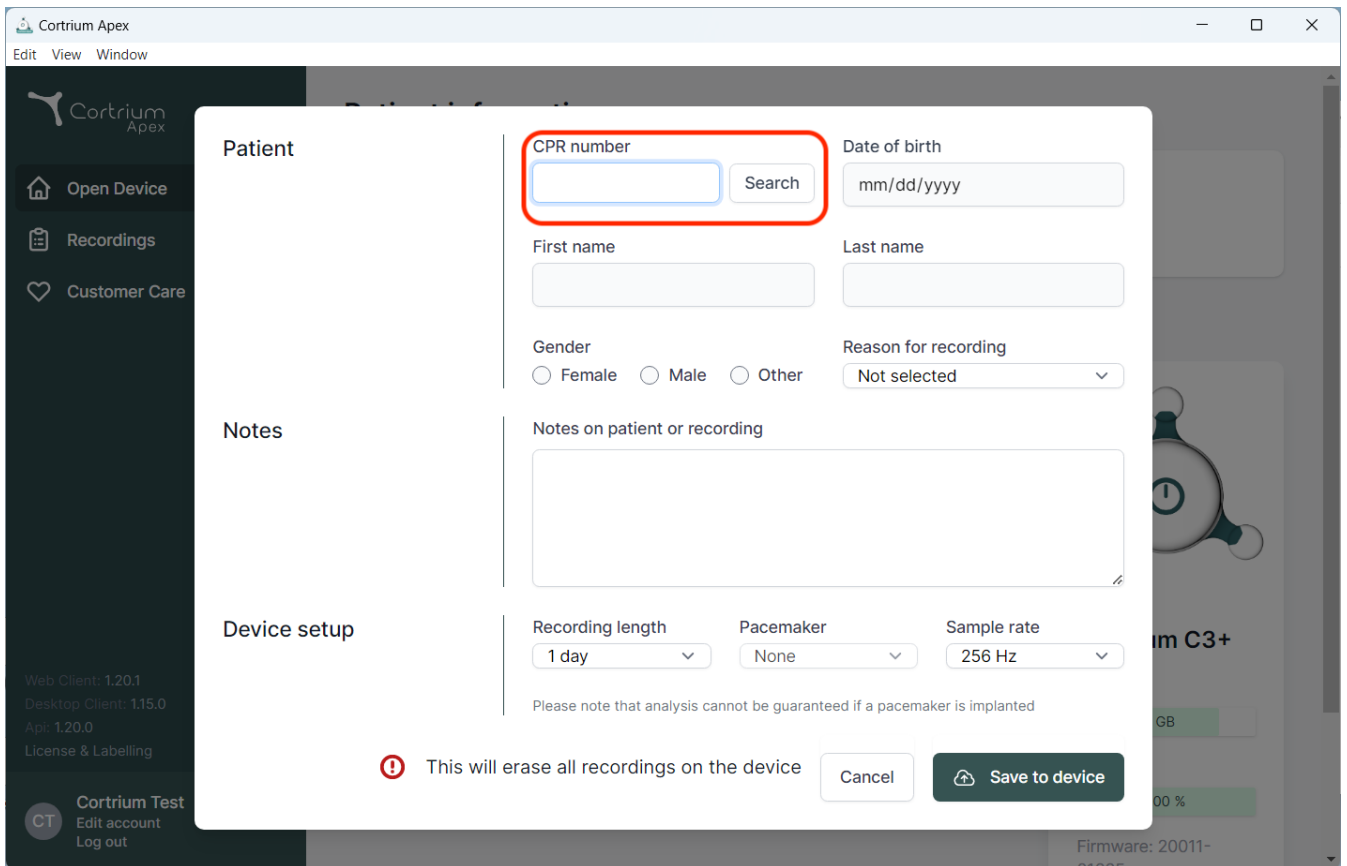
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot shows the Cortrium Apex web application interface. A modal dialog titled "Patient" is open, allowing for patient search and data entry. The dialog is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- CPR number:** A text input field is highlighted with a red box, with a "Search" button next to it.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

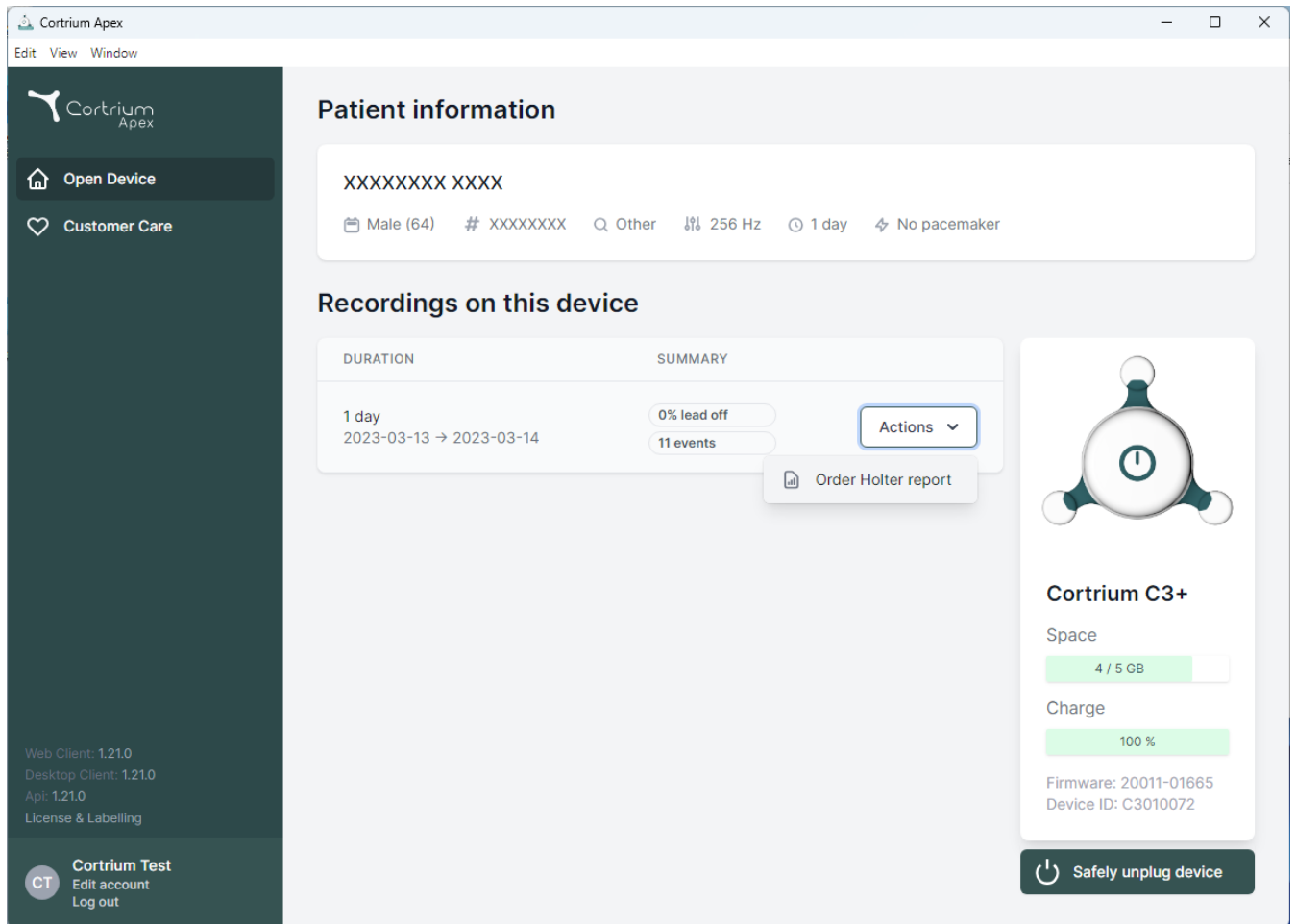
Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the dialog, there is a red warning icon and the text "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar includes version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0), license information, and user details for "Cortrium Test" (Edit account, Log out).

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), 'Customer Care' (with a heart icon), and version information: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar is a 'Cortrium Test' section with a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section contains a table with two columns: 'DURATION' and 'SUMMARY'. The table has one entry for '1 day' from '2023-03-13' to '2023-03-14', showing '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, showing the option 'Order Holter report'. On the right side, there is a device status card for 'Cortrium C3+'. It displays a battery icon, 'Space' (4 / 5 GB), 'Charge' (100 %), 'Firmware: 20011-01665', and 'Device ID: C3010072'. At the bottom of this card is a 'Safely unplug device' button with a power icon.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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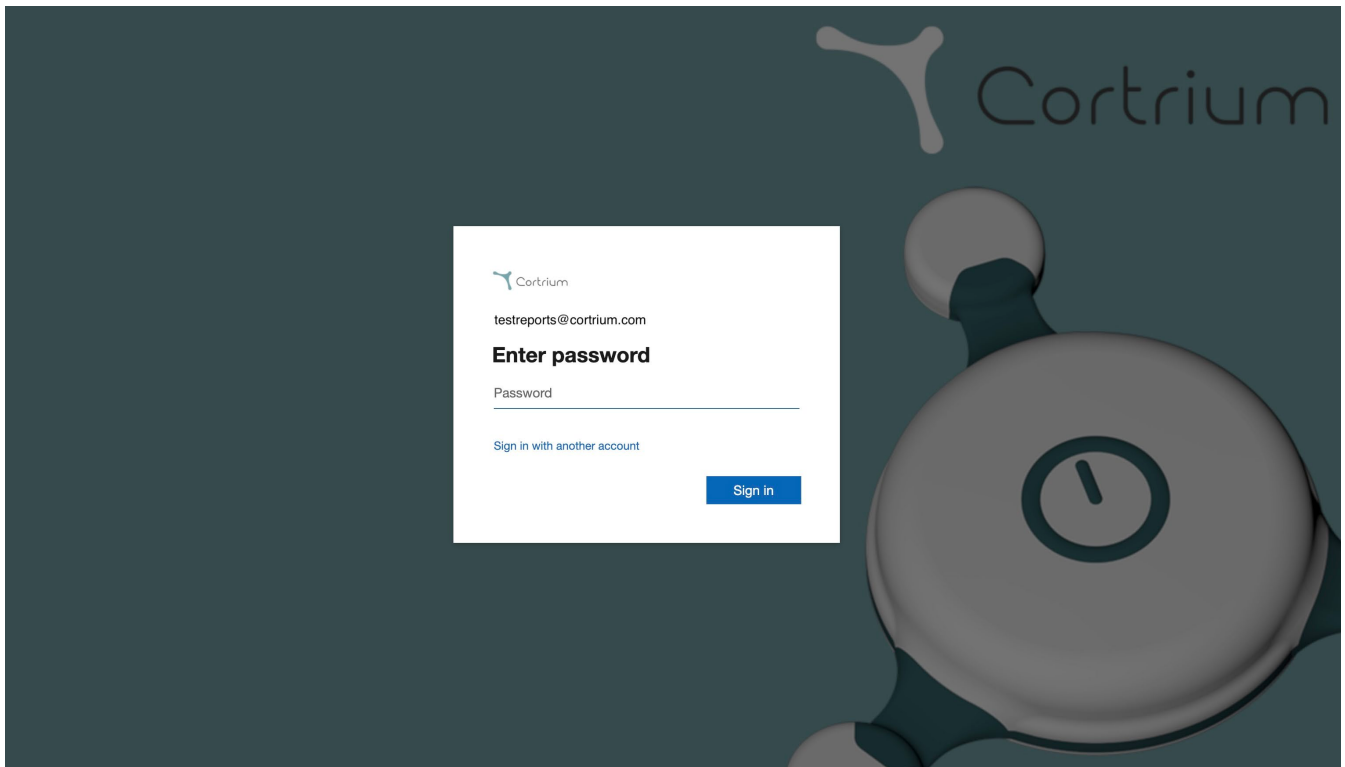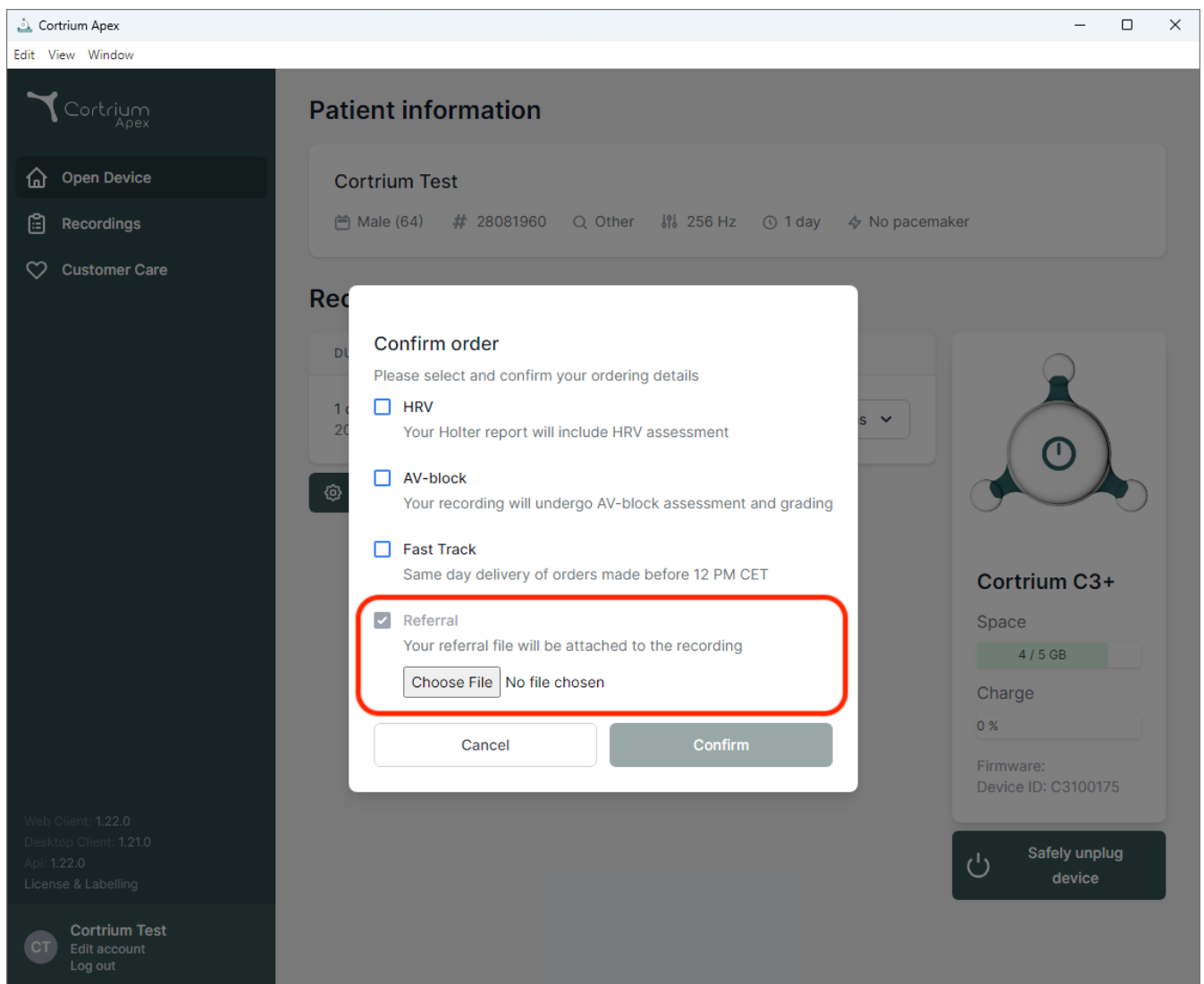
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
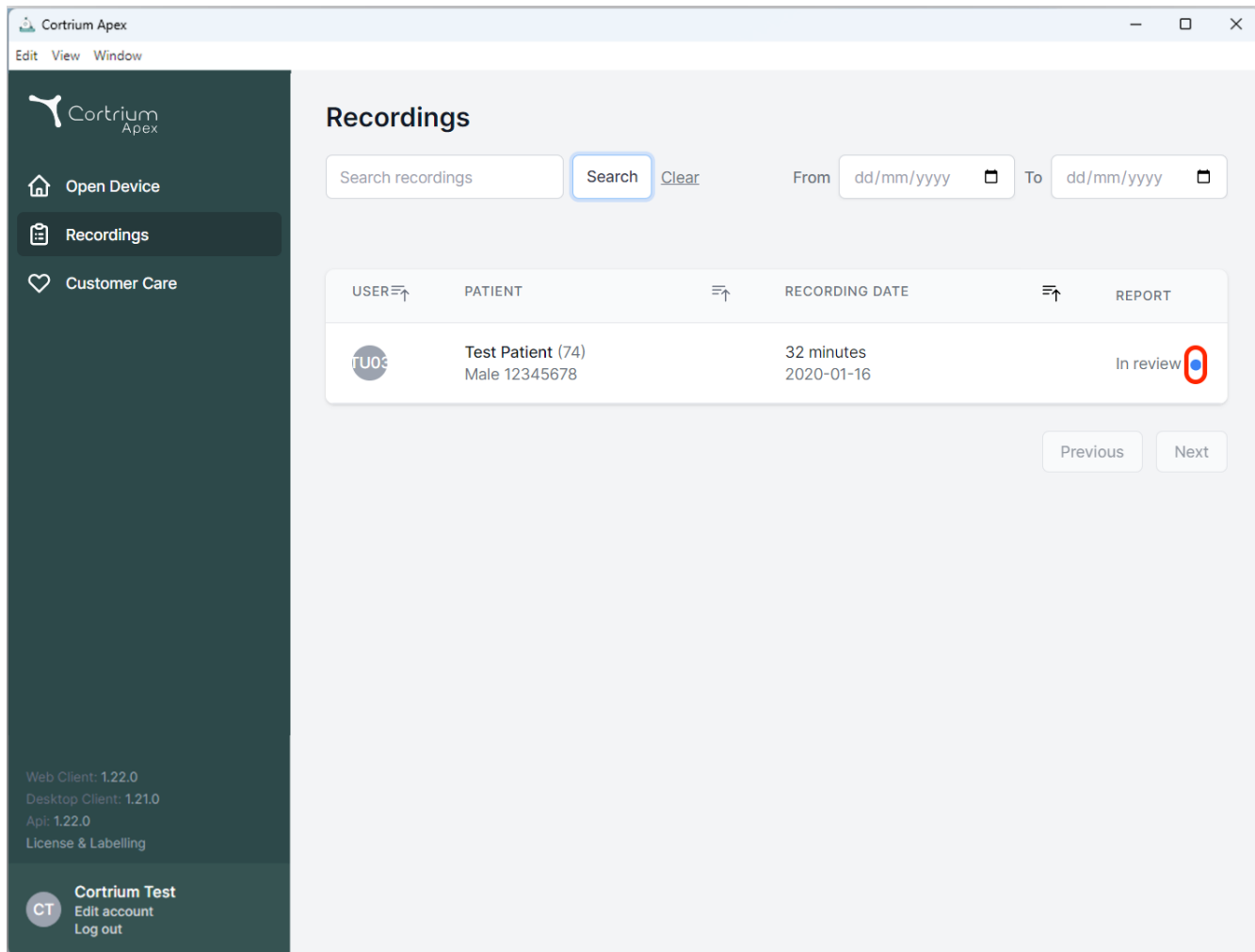
Search recordings [Clear](#)

From To

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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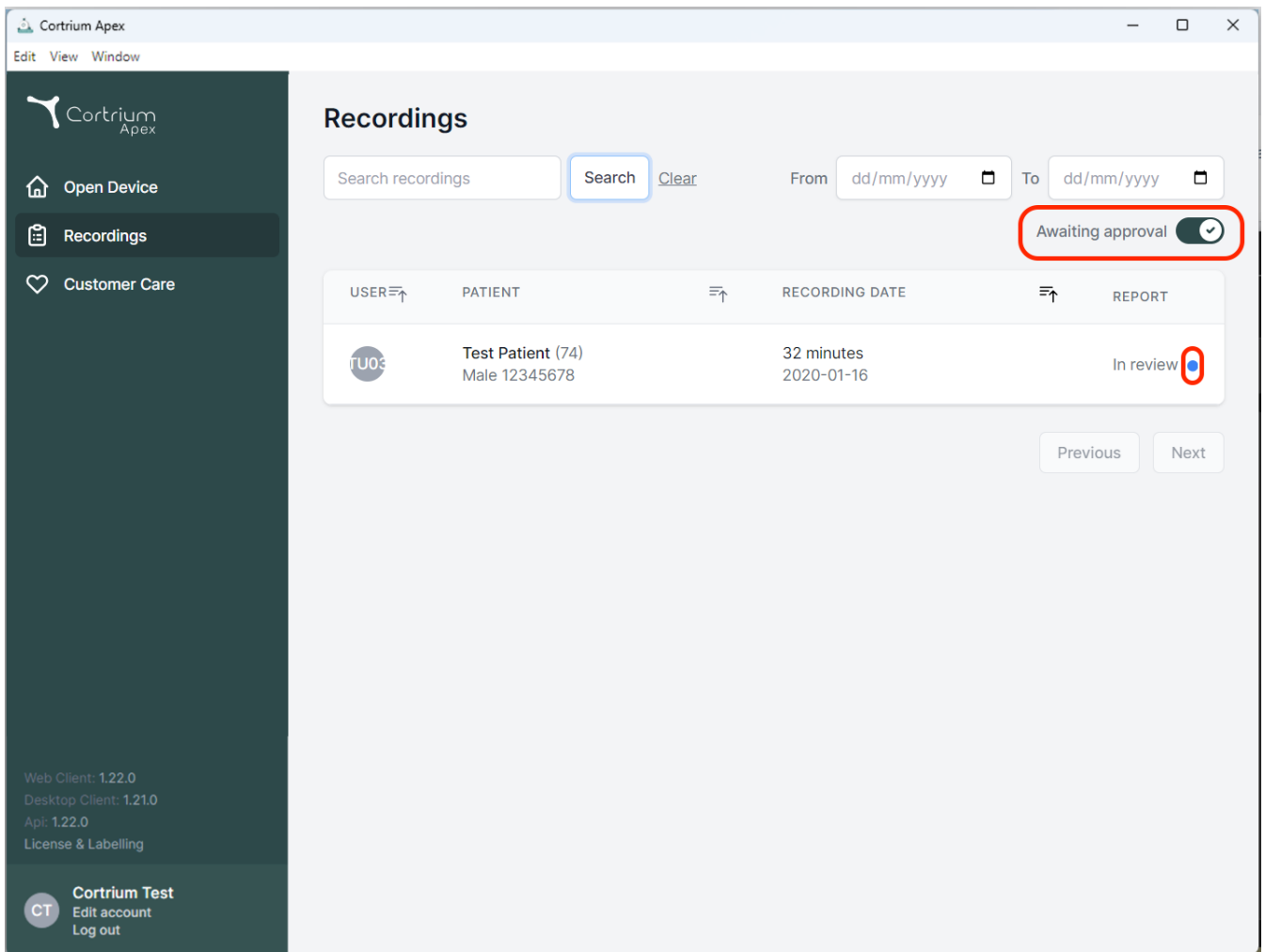Recordings

Search recordings From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
U03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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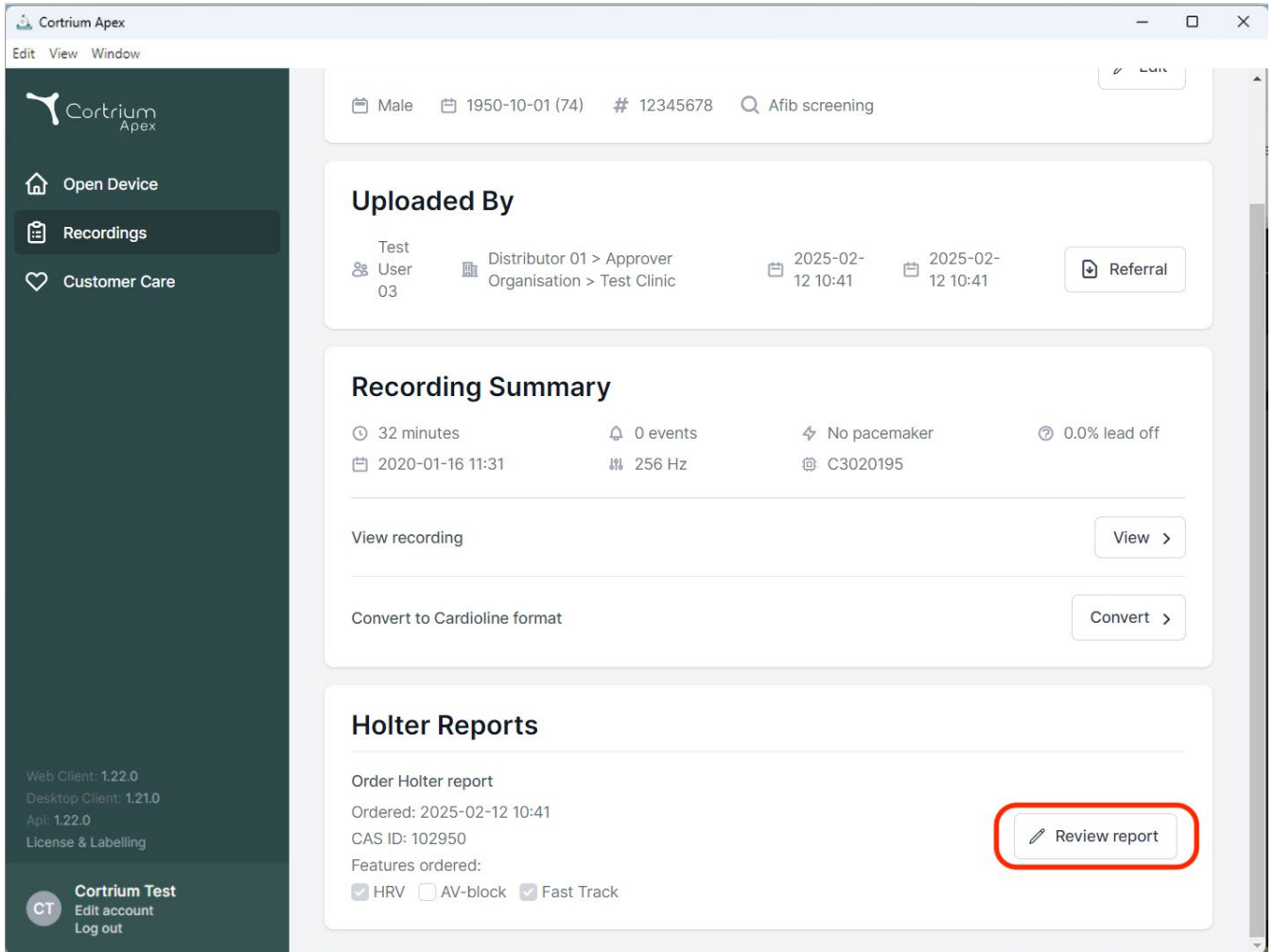The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, Afib screening. Below this is the "Uploaded By" section, showing Test User 03, Distributor 01 > Approver Organisation > Test Clinic, and upload dates 2025-02-12 10:41. A "Referral" button is present. The "Recording Summary" section displays: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes "View recording" and "Convert to Cardioline format" buttons. The "Holter Reports" section shows "Order Holter report", "Ordered: 2025-02-12 10:41", "CAS ID: 102950", and "Features ordered: HRV, AV-block, Fast Track". A "Review report" button is highlighted with a red rectangle.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

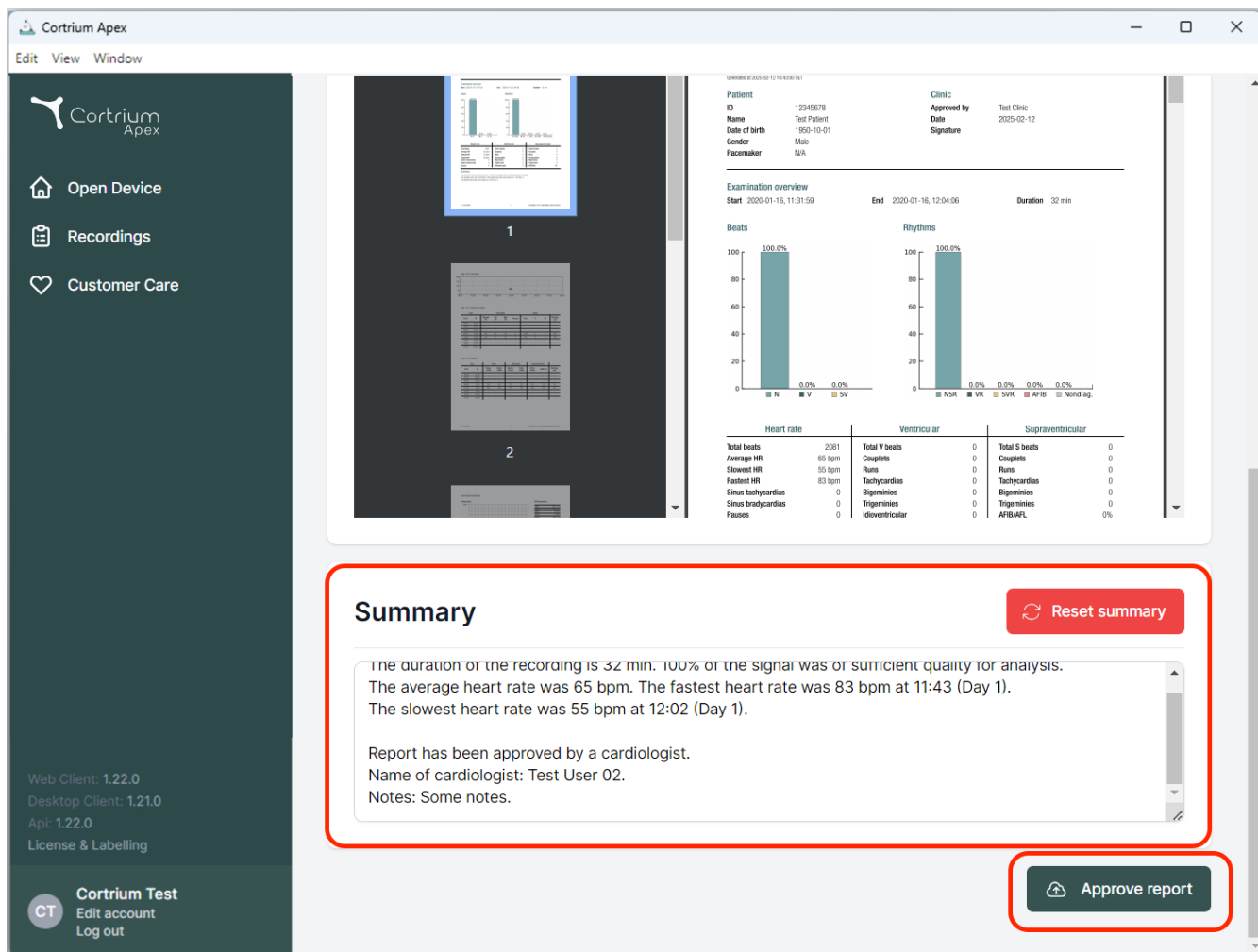
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

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