Cortrium Apex 1.26

Cortrium Apex 1.26

Lietošanas instrukcija

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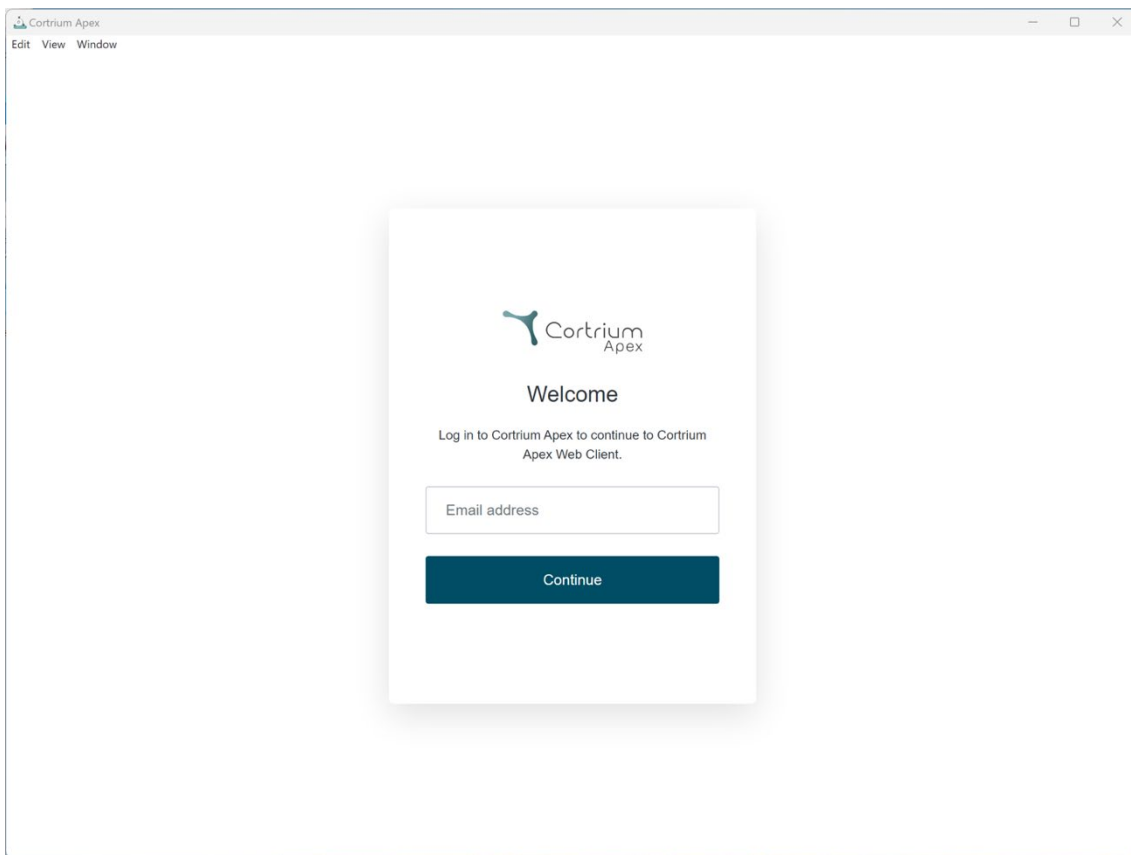
 Cortrium
Apex

Welcome

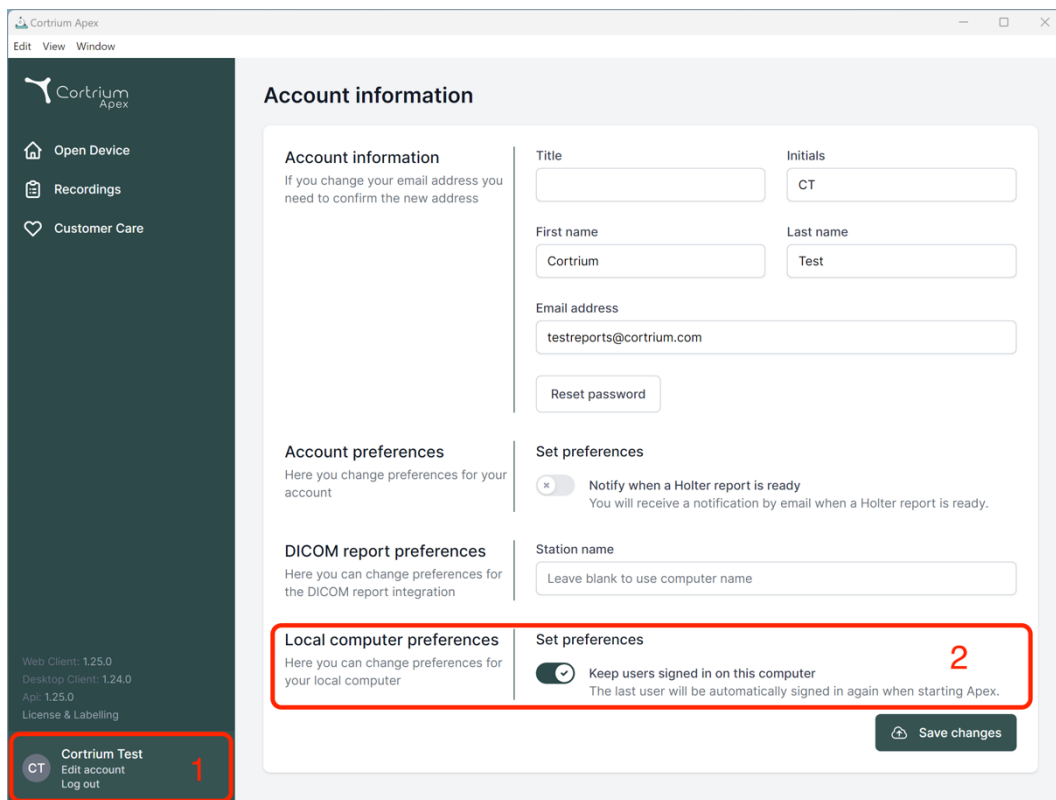
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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
The screenshot displays the Cortrium Apex application window. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, the user 'Cortrium Test' is logged in, with options to 'Edit account' and 'Log out' (marked with a red '1'). The main content area is titled 'Account information' and is divided into several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address), 'Account preferences' (with a toggle for notifications), 'DICOM report preferences' (with a field for Station name), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer' marked with a red '2'). A 'Save changes' button is located at the bottom right of the main content area.

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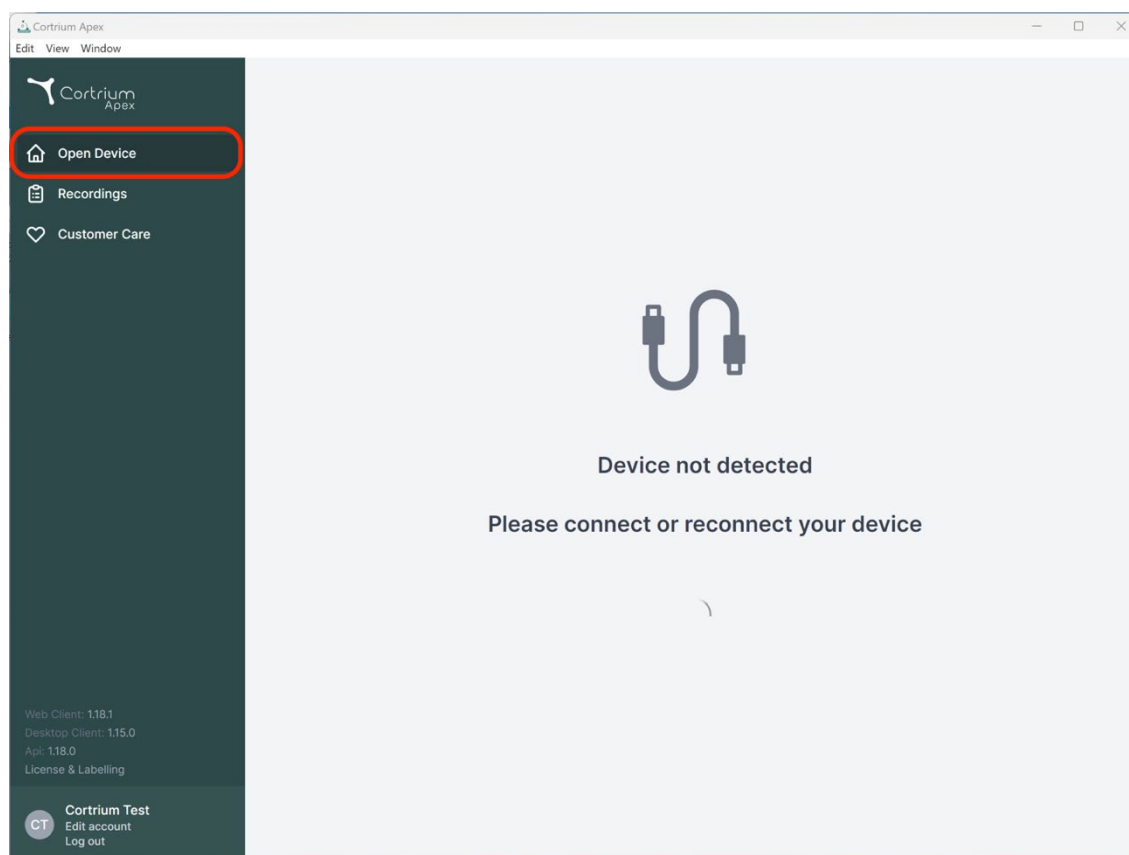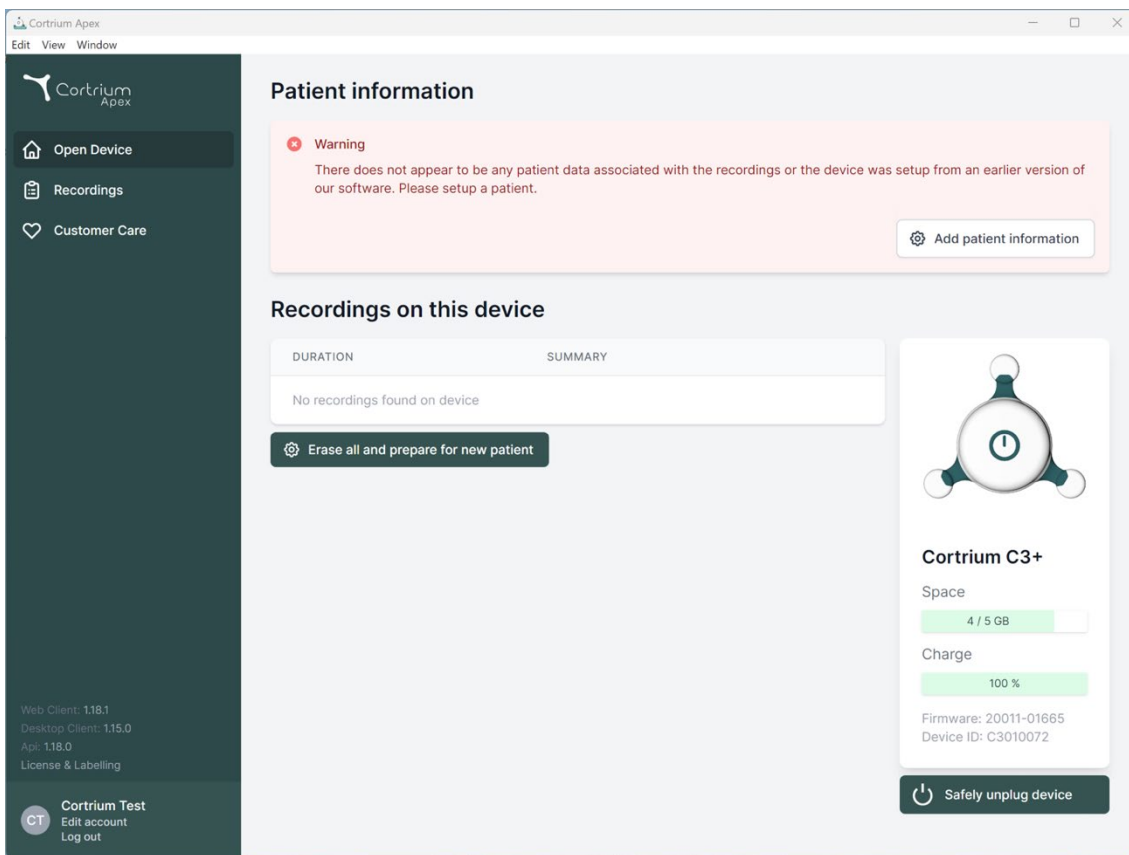
Patient information

Warning

There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.

[Add patient information](#)

Recordings on this device

DURATION	SUMMARY
No recordings found on device	

[Erase all and prepare for new patient](#)

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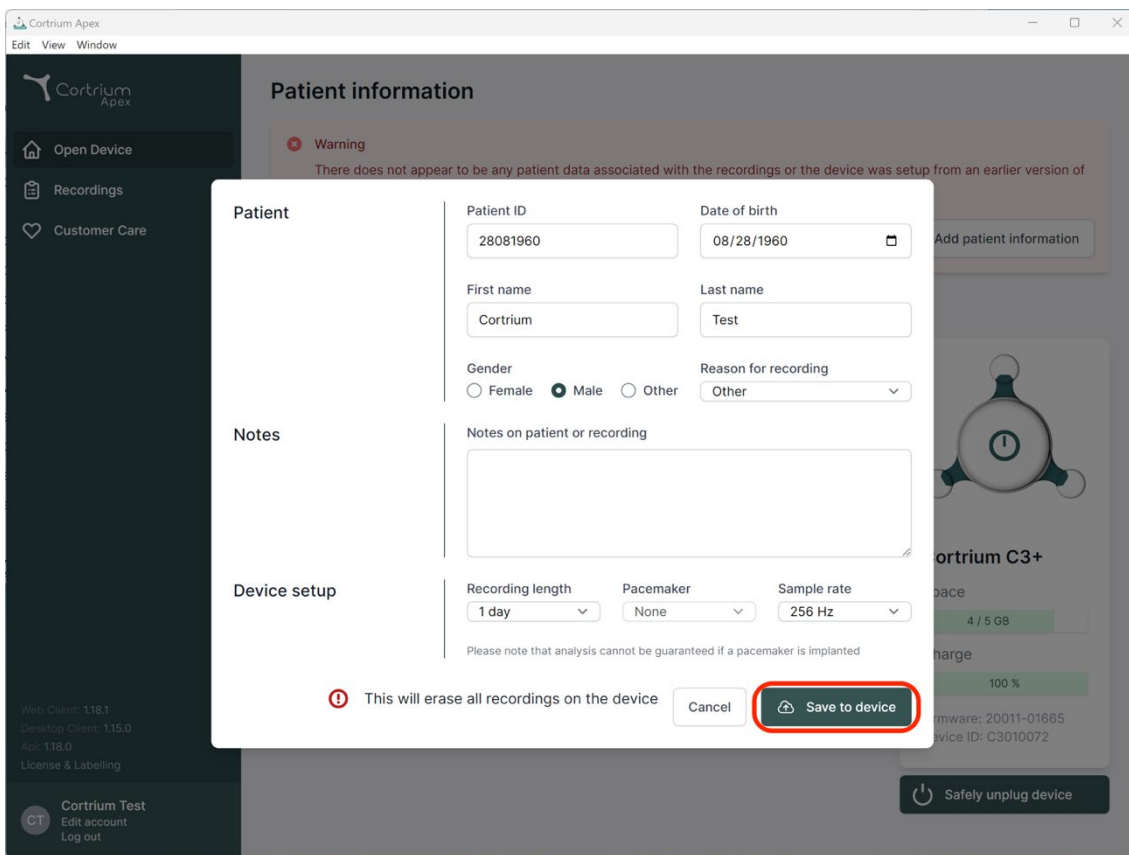
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: ☐ Female ☒ Male ☐ Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

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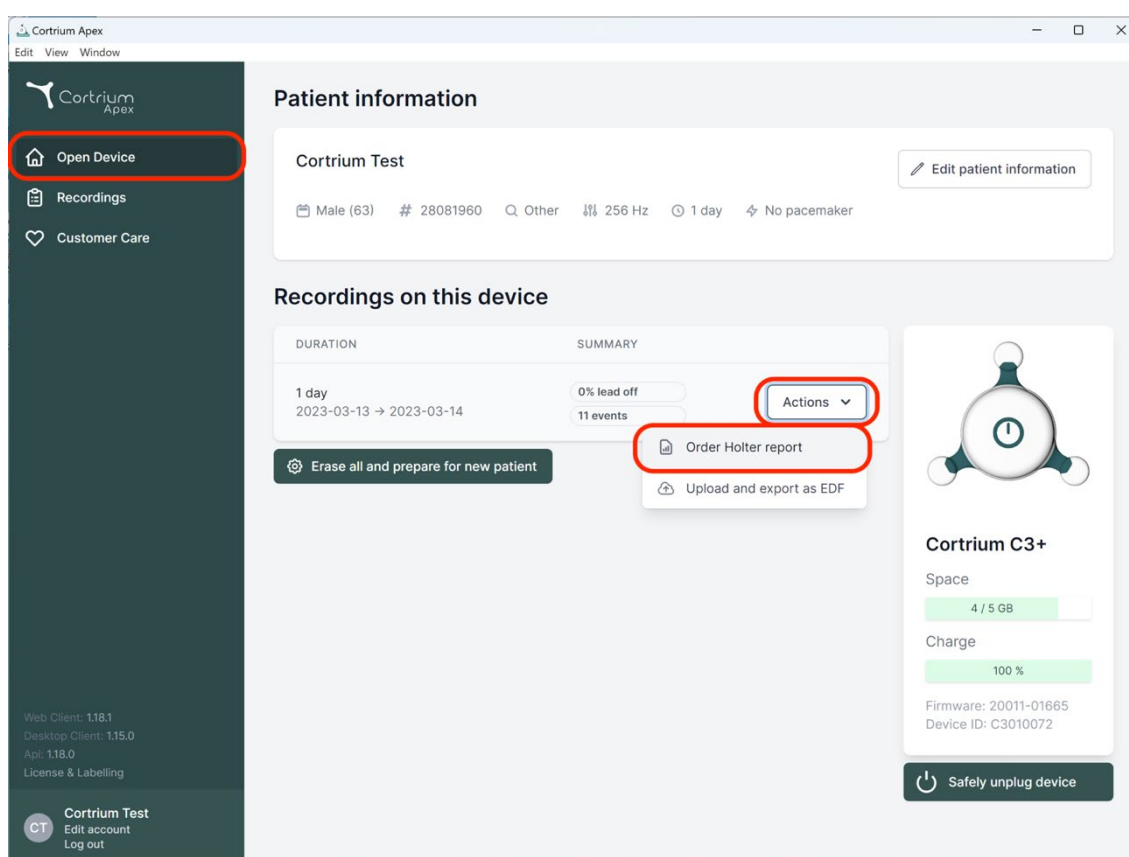
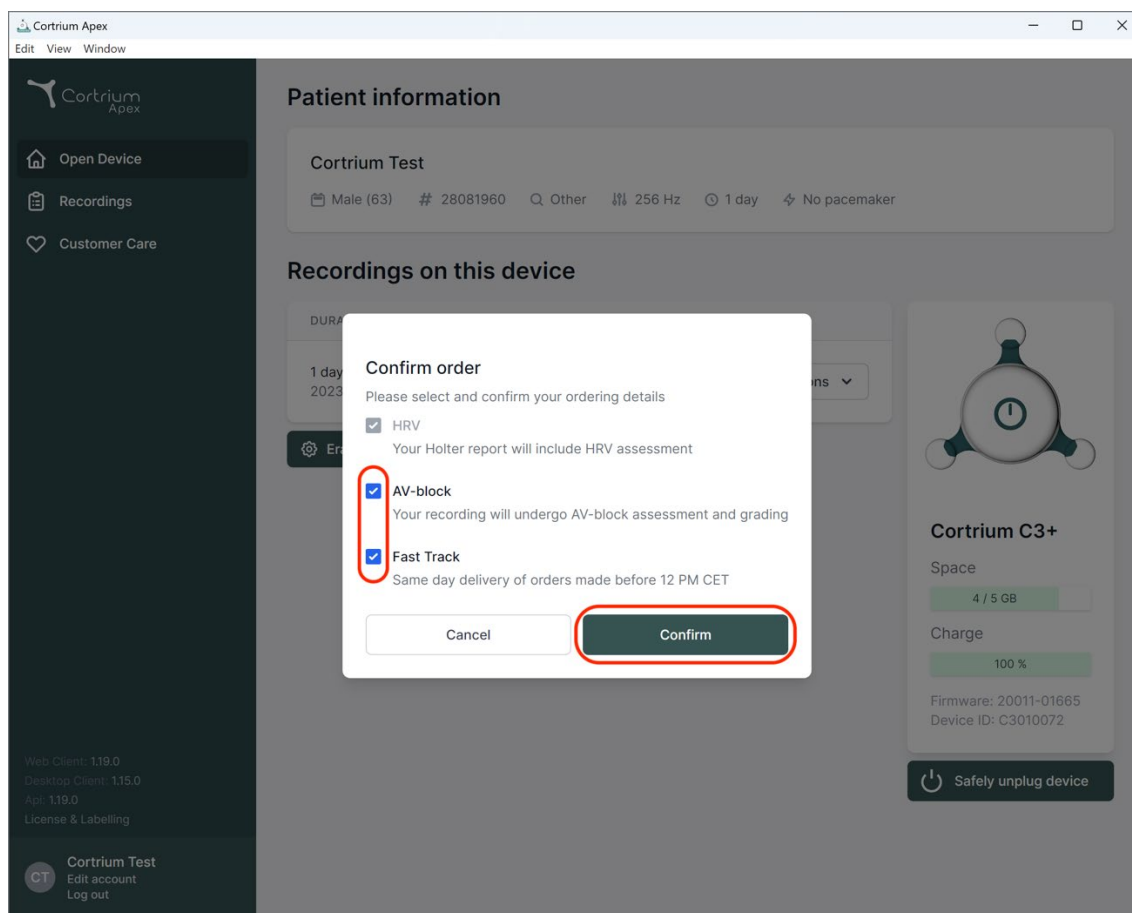
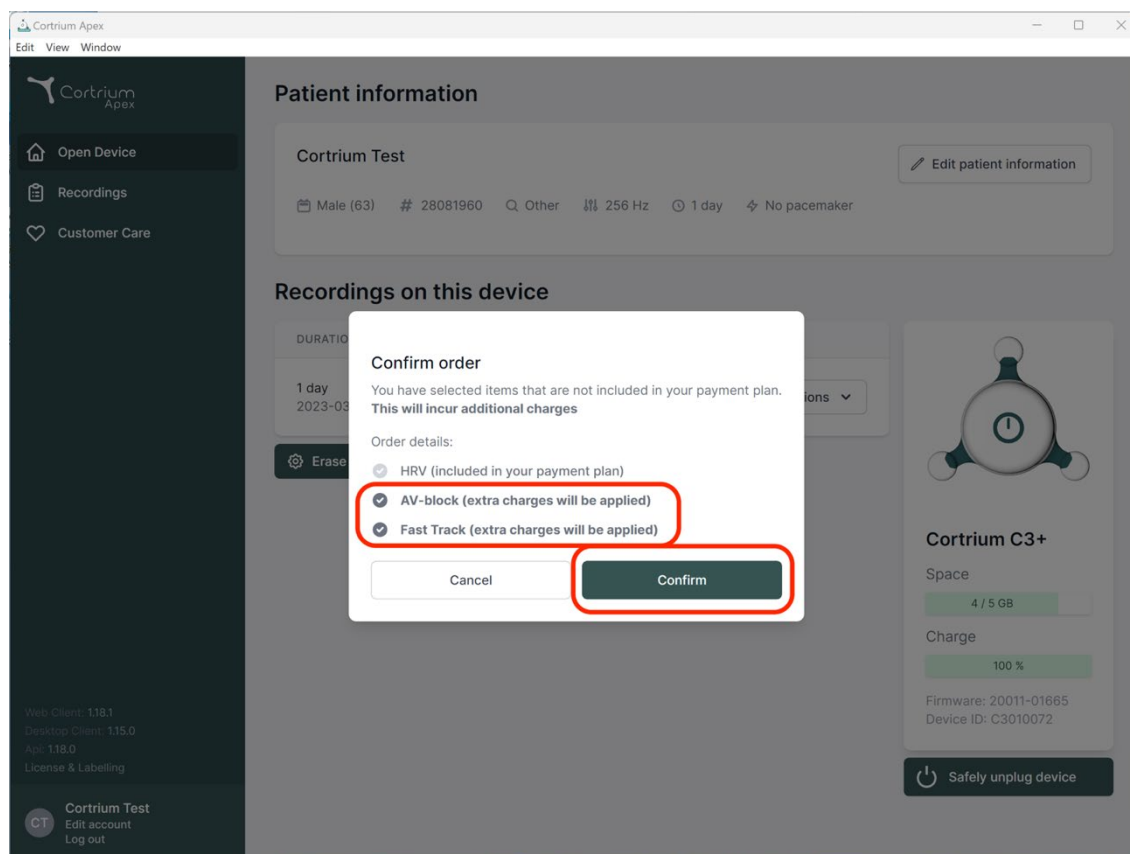
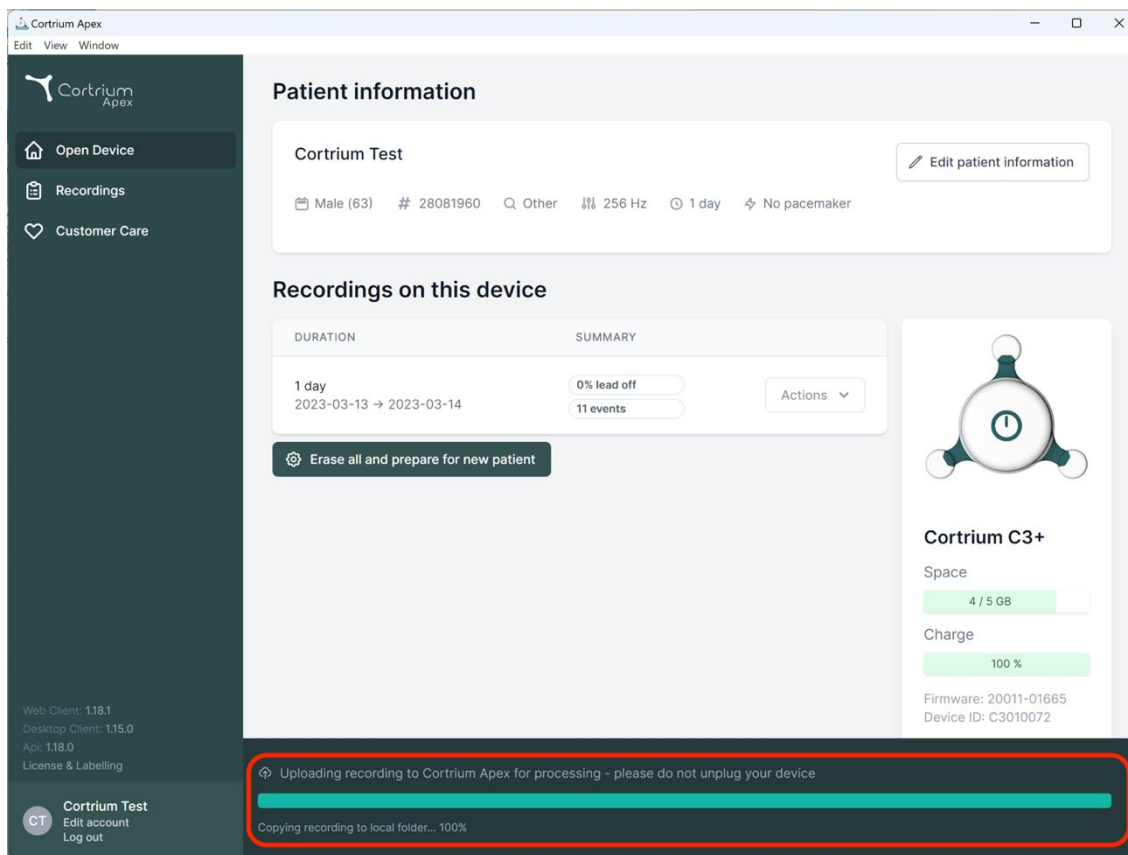
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and recording duration (1 day). Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom indicating 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. On the right, there is a device status card for 'Cortrium C3+' showing 4/5 GB space and 100% charge.

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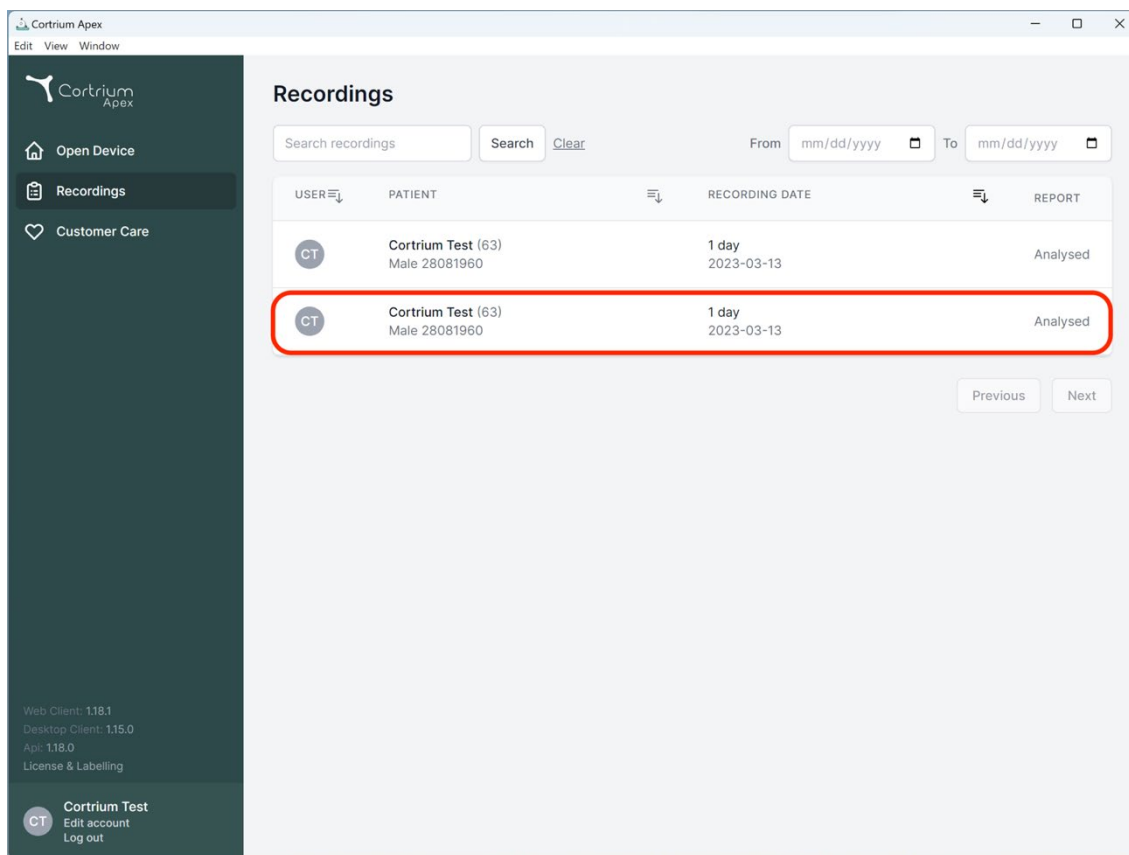
The screenshot displays the 'Recordings' section of the Cortrium Apex application. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area shows a table of recordings. At the top of the table are search and filter controls, including a 'Search recordings' input, 'Search' and 'Clear' buttons, and date range selectors for 'From' and 'To'. The table has four columns: 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. Two rows are visible, both identical, with the second row highlighted by a red border. Each row shows a user icon 'CT', patient name 'Cortrium Test (63)' and ID 'Male 28081960', a recording date of '1 day' and '2023-03-13', and a report status of 'Analysed'. At the bottom right of the table are 'Previous' and 'Next' navigation buttons. The bottom of the sidebar contains version information (Web Client: 1.18.1, Desktop Client: 1.15.0, API: 1.18.0, License & Labelling) and a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT

Cortrium Test

Edit account
Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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View recording View >

Download recording in .EDF format Download

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

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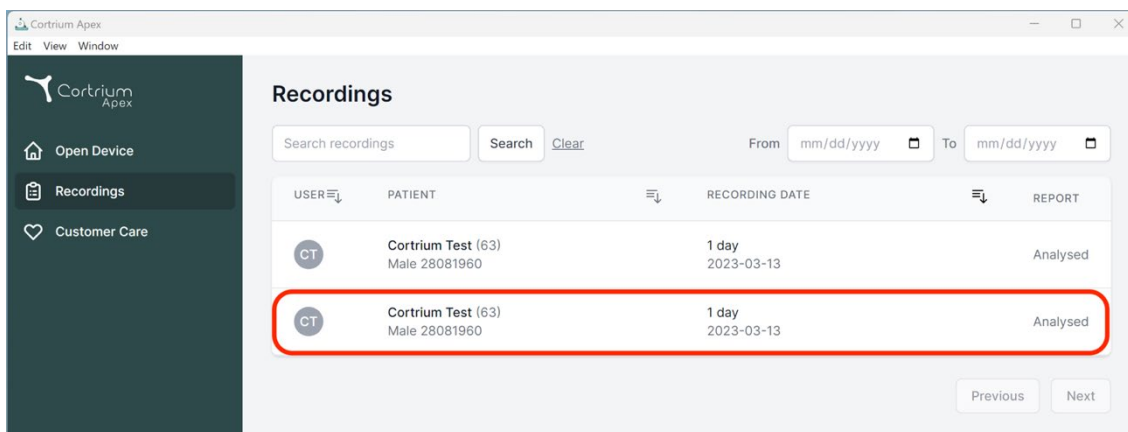
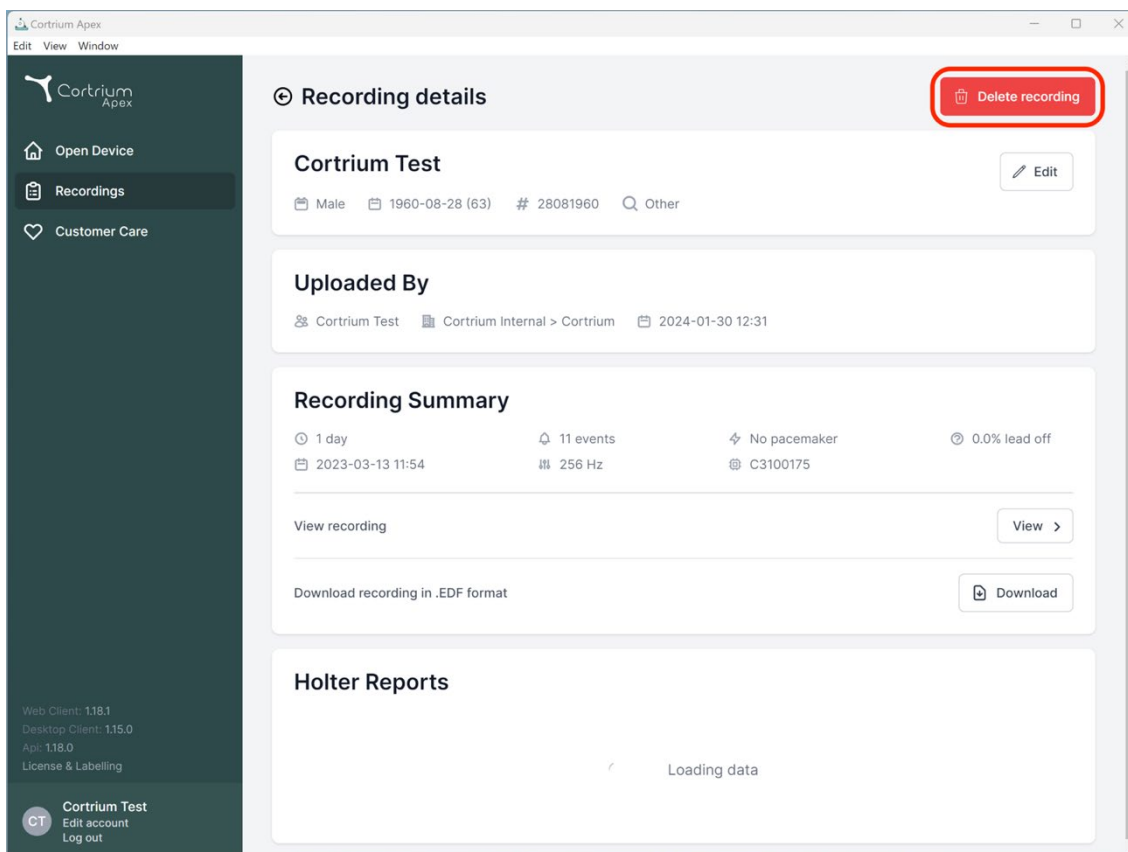
[View recording](#) View >

[Download recording in .EDF format](#) Download

Holter Reports

Loading data

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

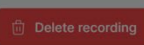
 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

 Delete recording

Cortrium Test

 Male  1960-08-28 (63)  # 28081960  Other  Edit

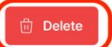
Uploaded By


 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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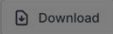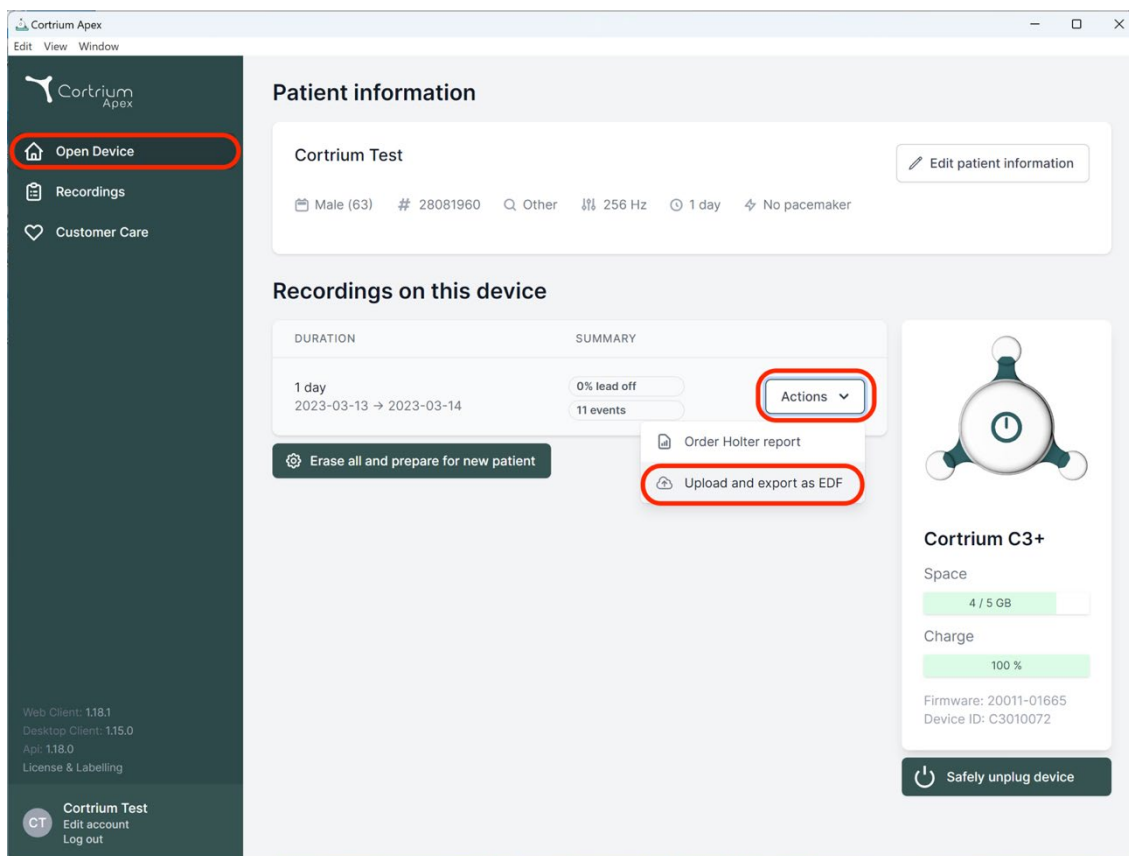
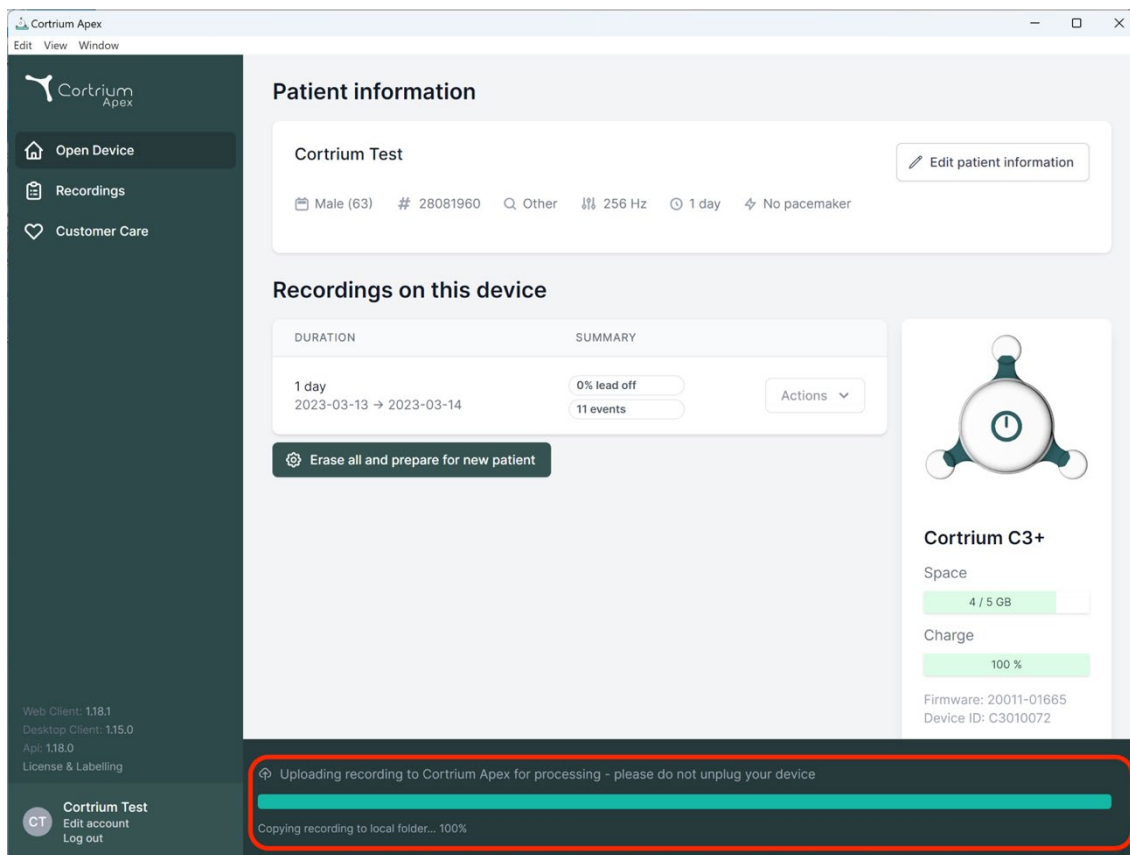
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

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Charge: 100 %

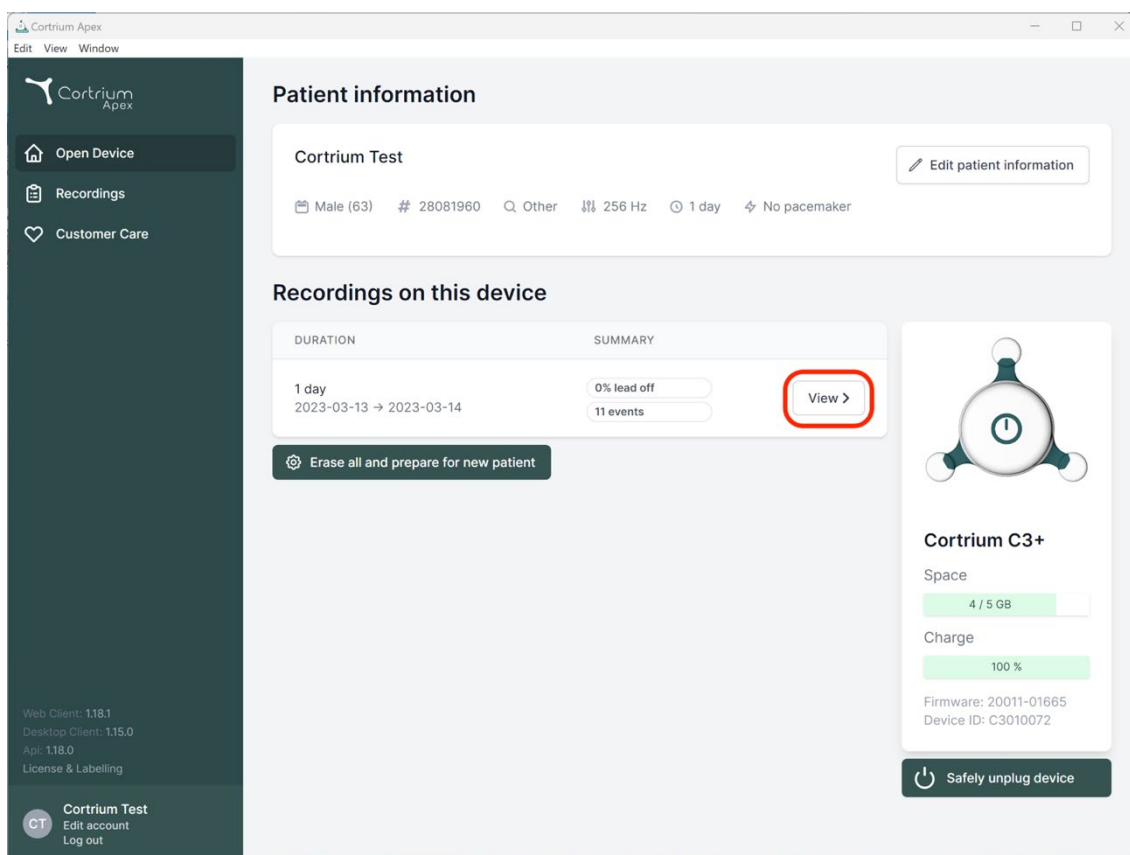
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

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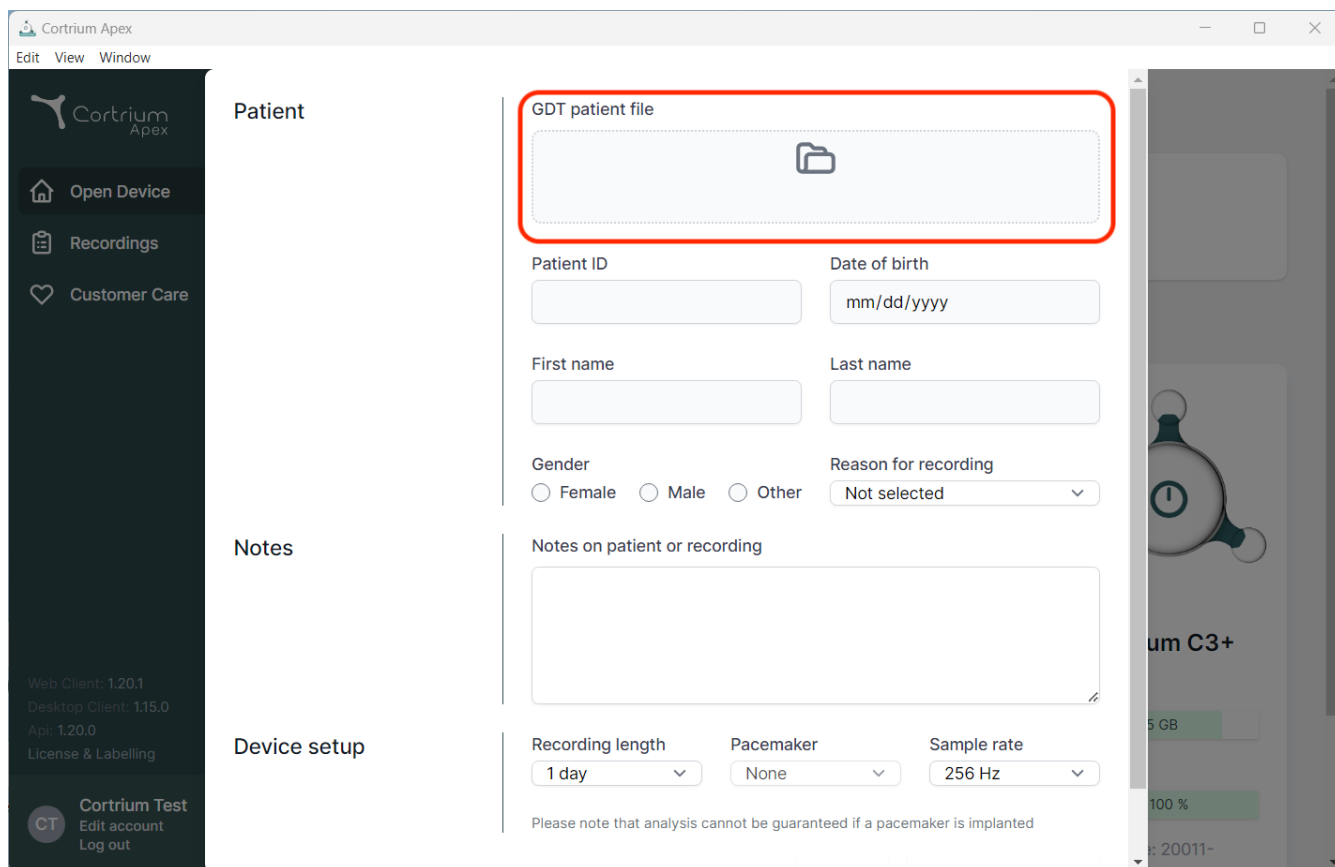
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

Patient section:

- GDT patient file:** A red-bordered box containing a folder icon, indicating where to upload the patient file.
- Patient ID:** A text input field.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes section:

- Notes on patient or recording:** A large text area for entering notes.

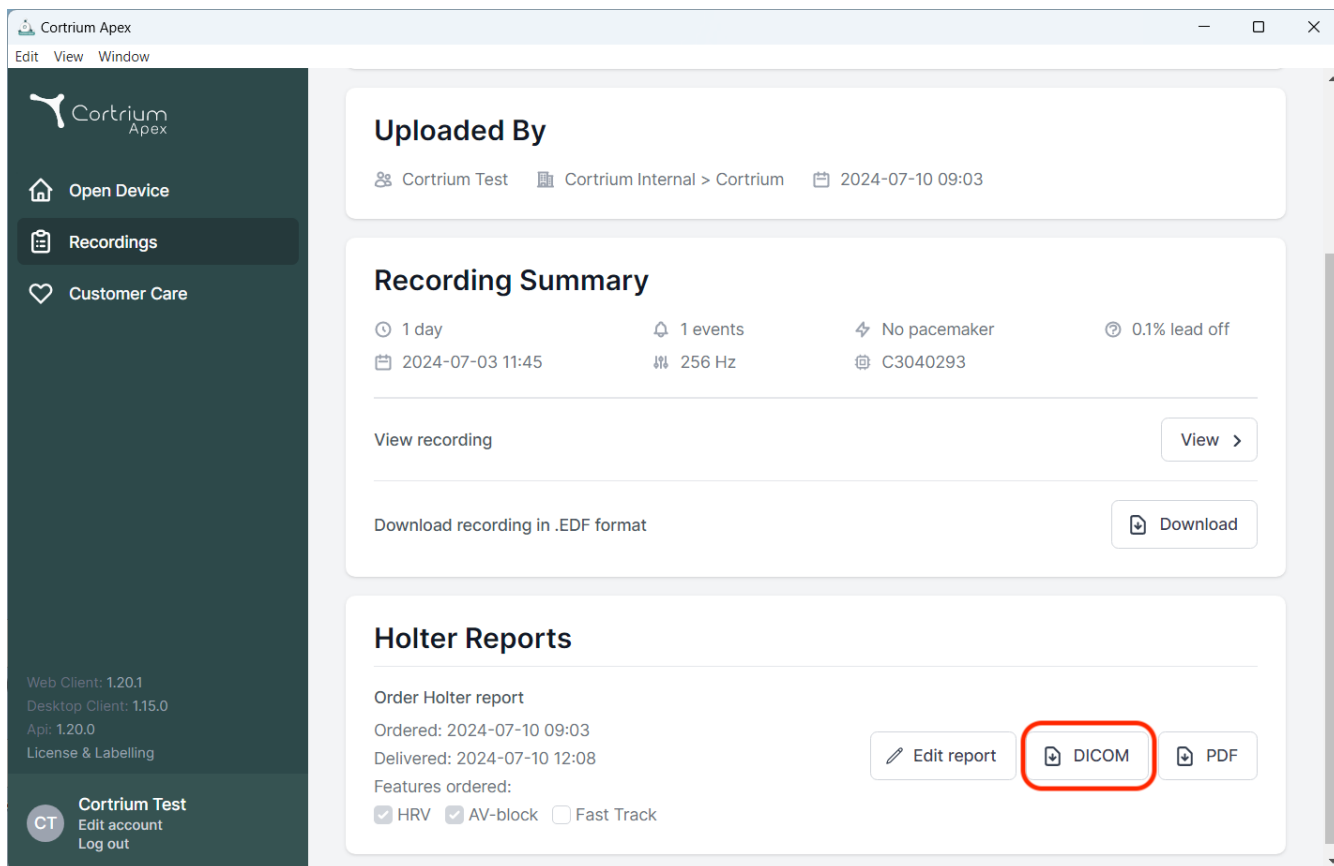
Device setup section:

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

Below the device setup section, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom left of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

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Account information

Account information

If you change your email address you need to confirm the new address

Title

Initials

First name

Last name

Email address

Reset password

Account preferences

Here you change preferences for your account

Set preferences

☐

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

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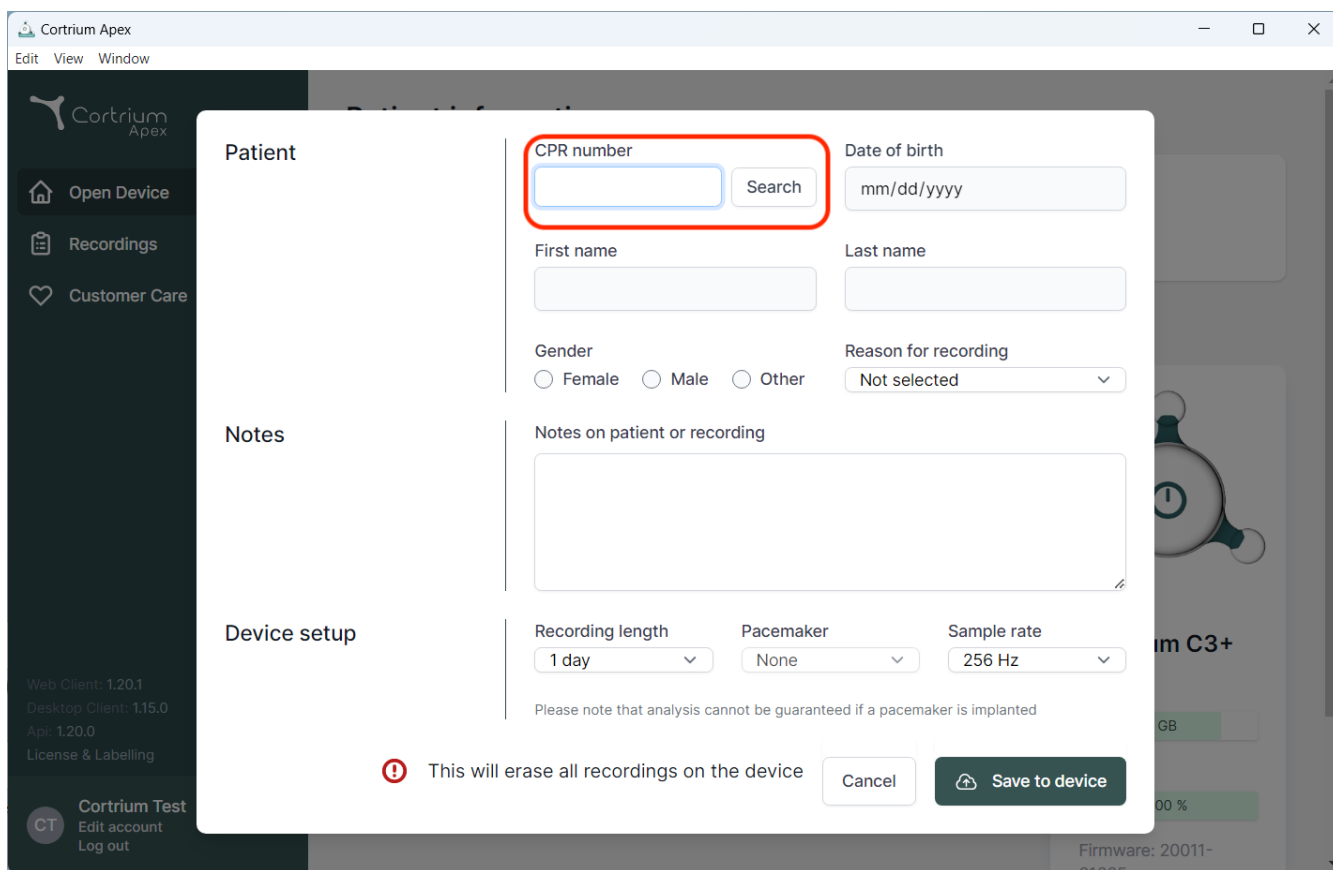
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for entering patient information. The "CPR number" field is highlighted with a red rectangle. The form includes fields for "Date of birth", "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), "Reason for recording" (a dropdown menu), and "Notes on patient or recording" (a text area). Below the form, there is a "Device setup" section with dropdown menus for "Recording length", "Pacemaker", and "Sample rate". A warning message at the bottom states: "This will erase all recordings on the device". The background shows the Cortrium Apex sidebar with options like "Open Device", "Recordings", and "Customer Care".

Patient

CPR number Search

Date of birth

First name

Last name

Gender
☐ Female ☐ Male ☐ Other

Reason for recording
Not selected

Notes

Notes on patient or recording

Device setup

Recording length Pacemaker Sample rate

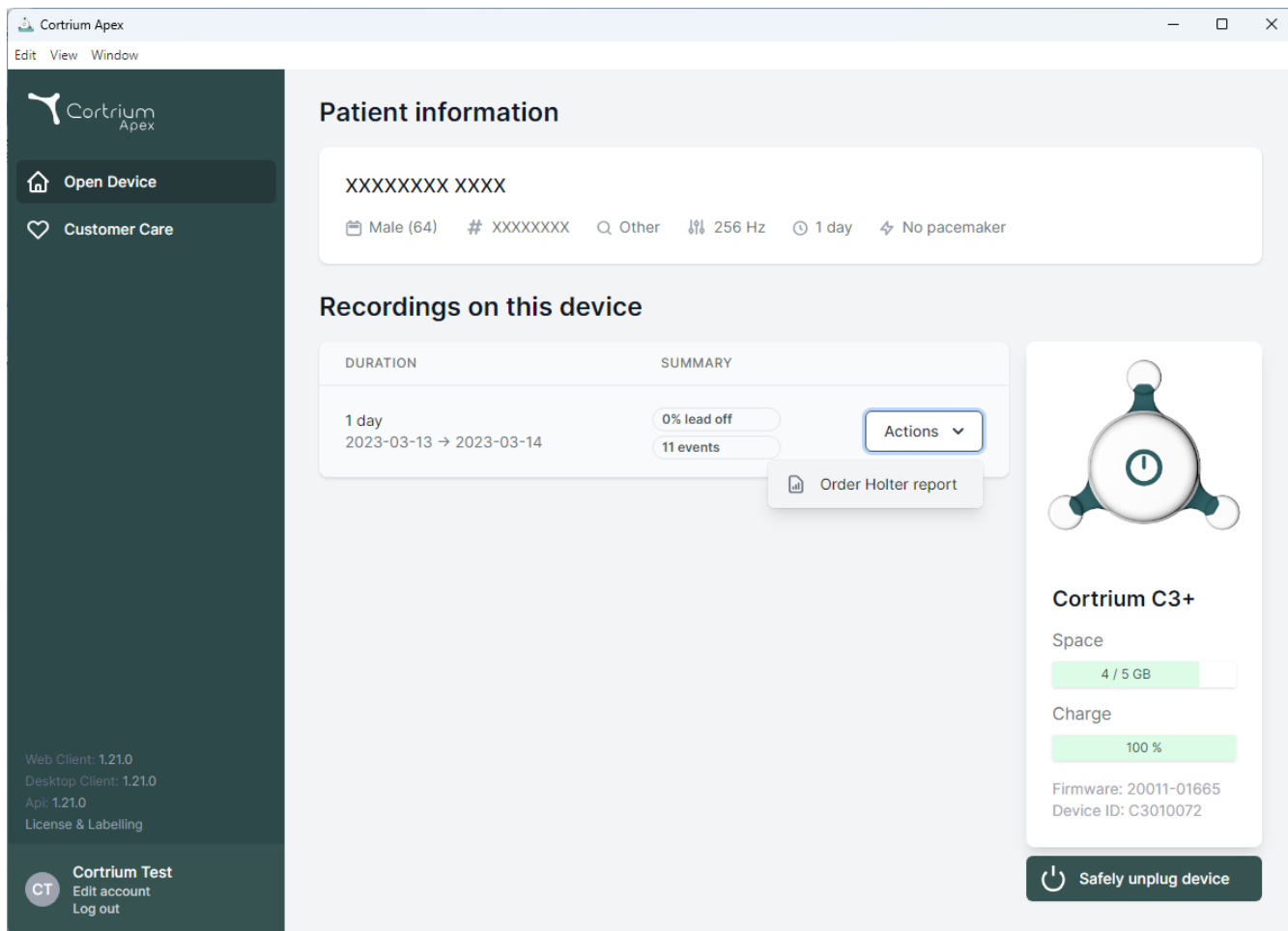
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo, 'Open Device', and 'Customer Care' links. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

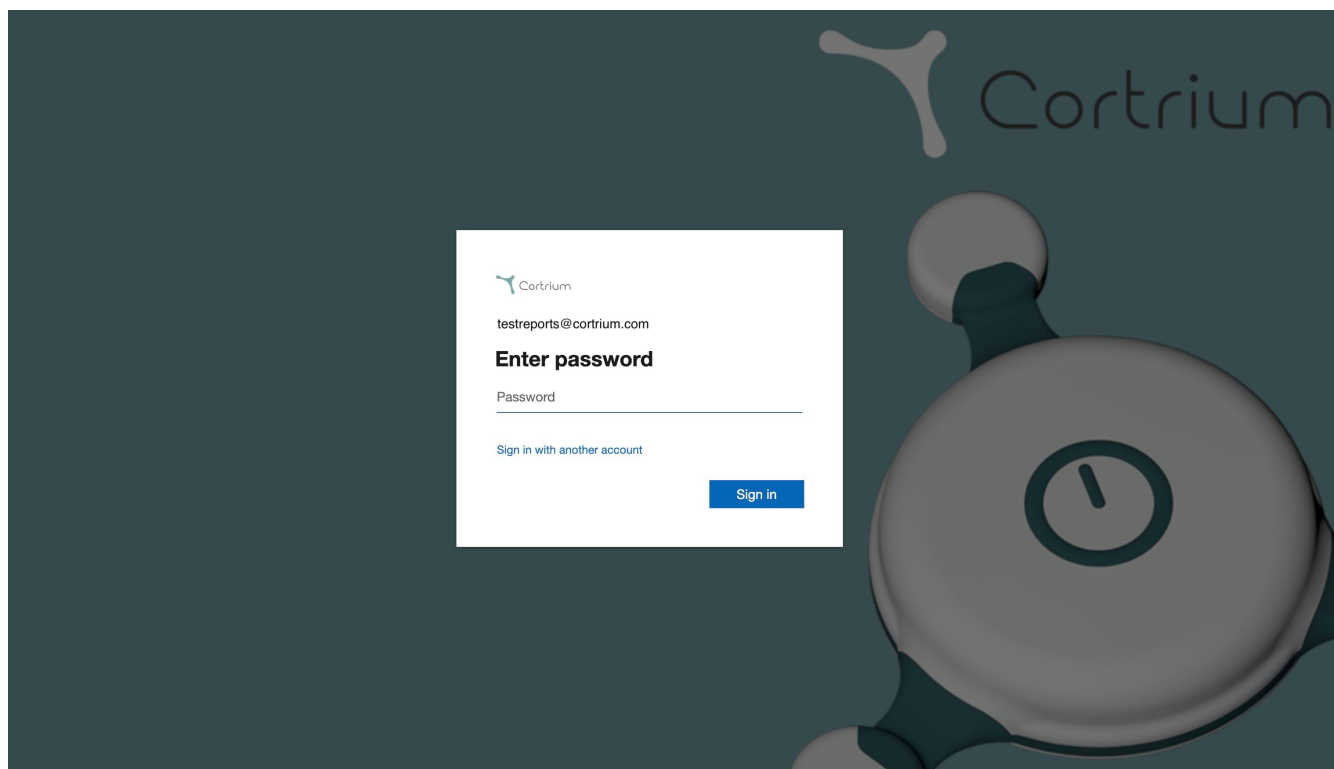
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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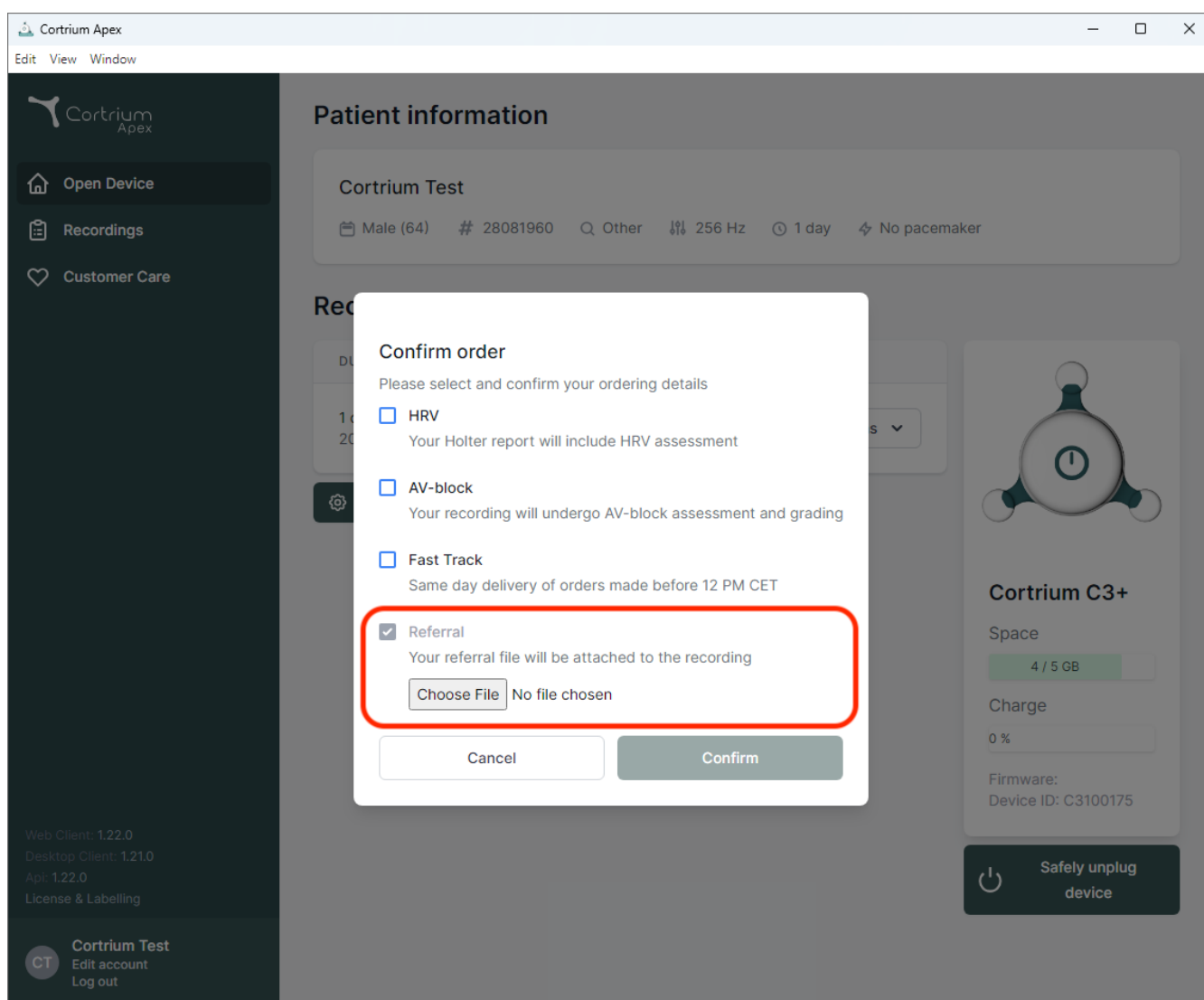
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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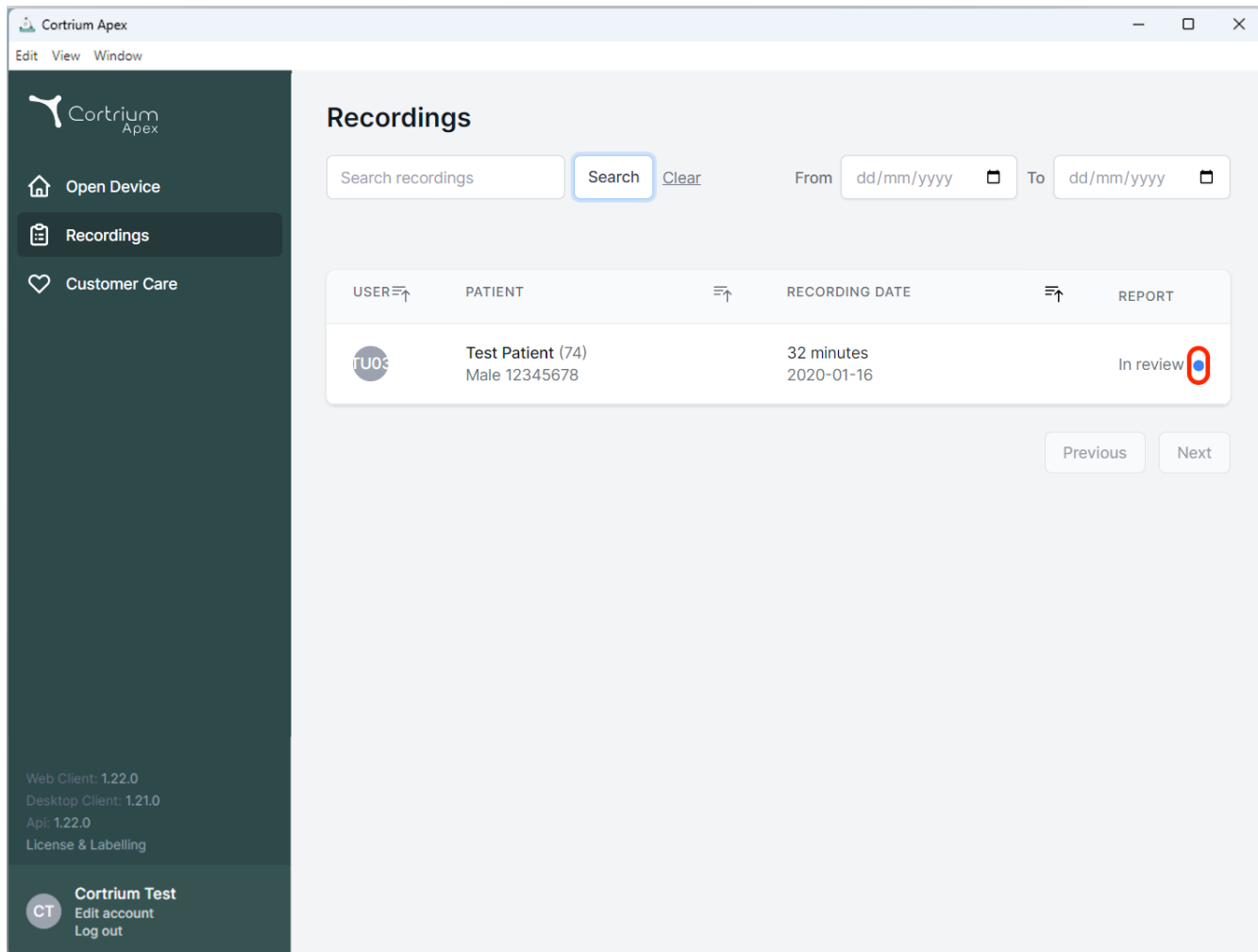
Recordings

Search recordings [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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
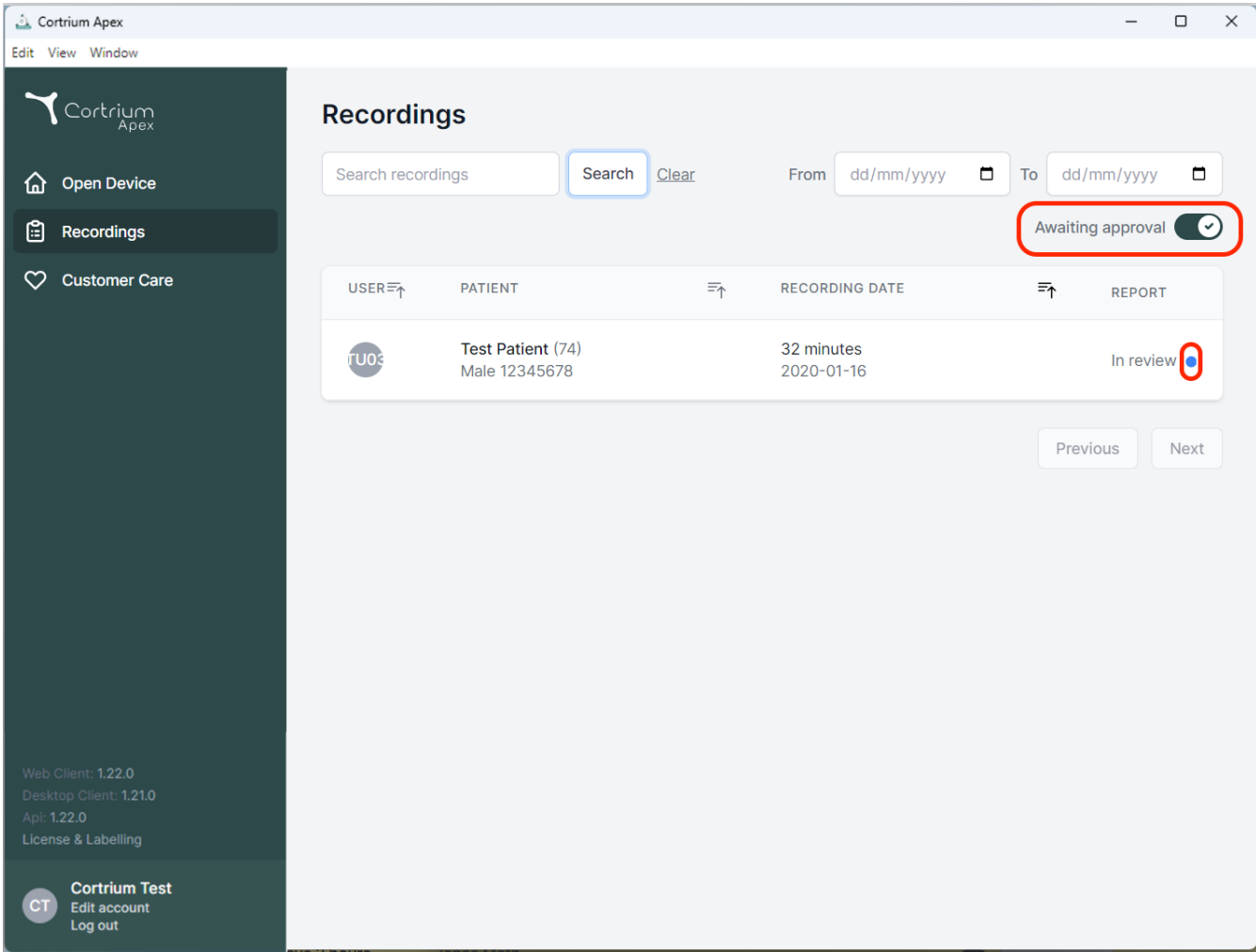
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: "Open Device", "Recordings" (highlighted), and "Customer Care". At the bottom of the sidebar, it shows version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling) and a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

The main content area is titled "Recordings". It features a search bar with a "Search" button and a "Clear" link. To the right of the search bar are date pickers for "From" and "To" in "dd/mm/yyyy" format. A toggle switch labeled "Awaiting approval" is set to "On" and is circled in red. Below these elements is a table with the following columns: USER, PATIENT, RECORDING DATE, and REPORT.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●


At the bottom right of the table area are "Previous" and "Next" buttons. The "In review" status in the table is also circled in red.

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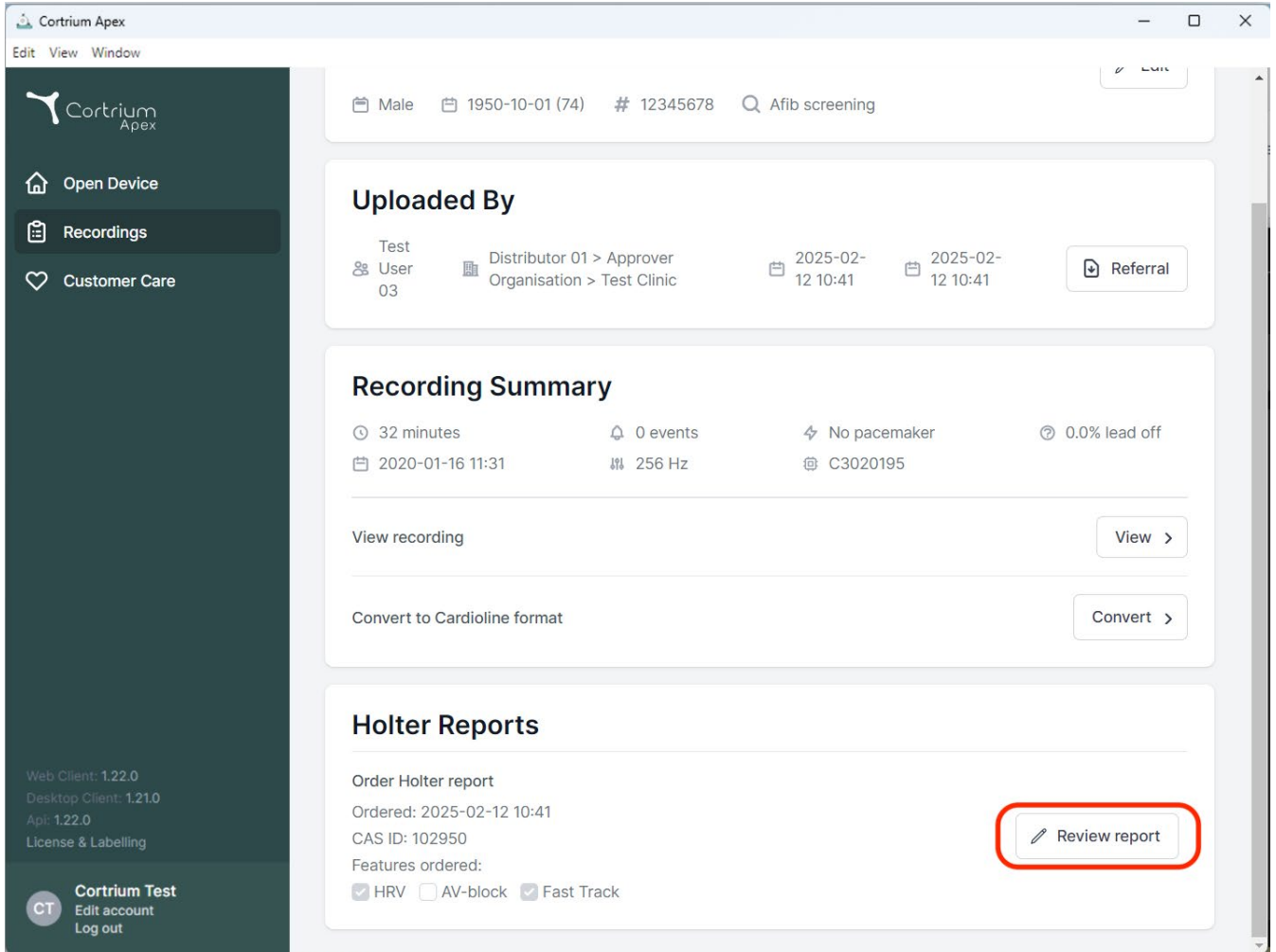The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and '2025-02-12 10:41', and a 'Referral' button. The 'Recording Summary' section displays: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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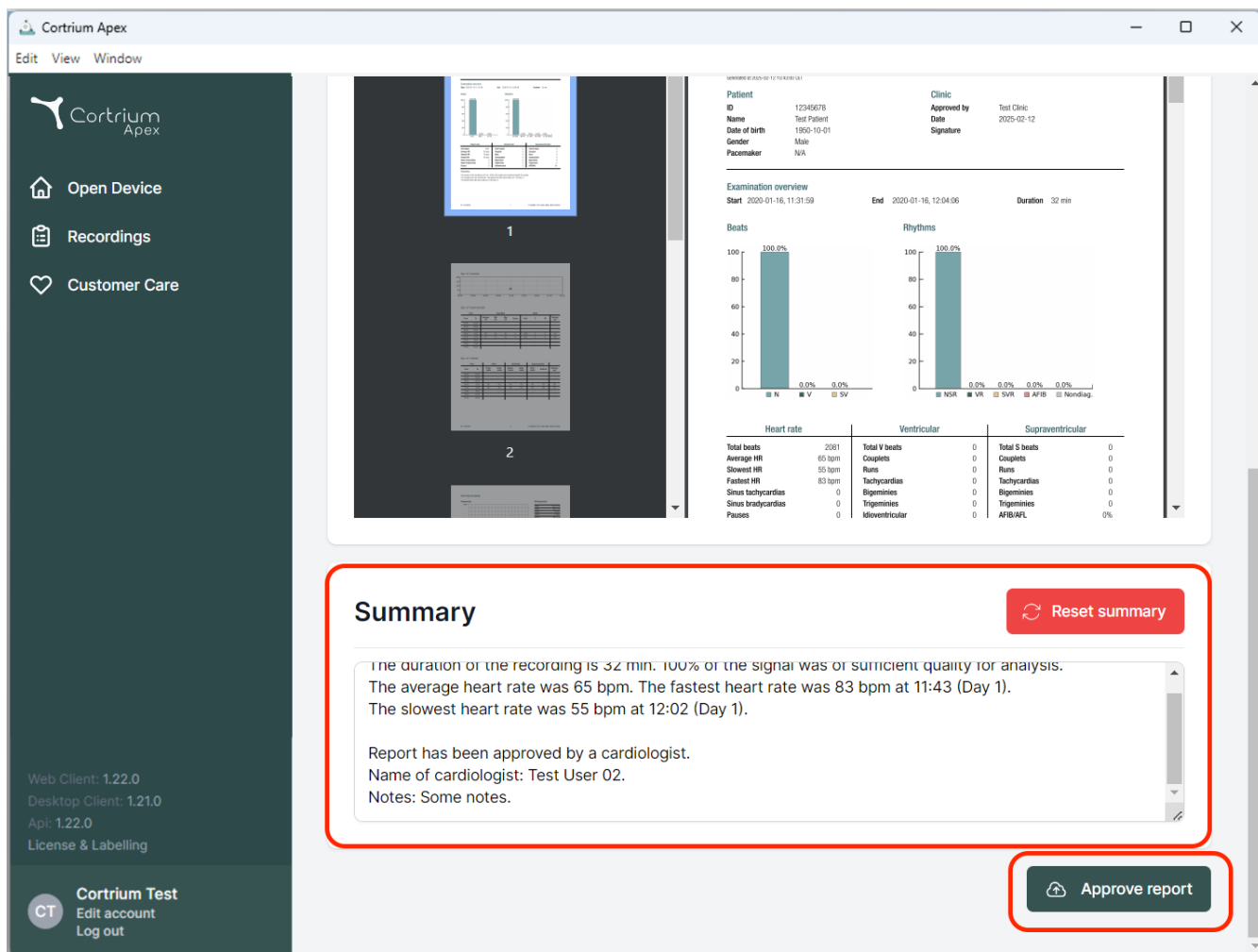Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient
ID: 12345678
Name: Test Patient
Date of birth: 1950-10-01
Gender: Male
Pacemaker: N/A

Clinic
Approved by: Test Clinic
Date: 2025-02-12
Signature:

Examination overview
Start: 2020-01-16, 11:31:59
End: 2020-01-16, 12:04:06
Duration: 32 min

Beats
100.0%
0.0% 0.0% 0.0%

Rhythms
100.0%
0.0% 0.0% 0.0% 0.0% 0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AFIB/AF	0%

Summary Reset summary

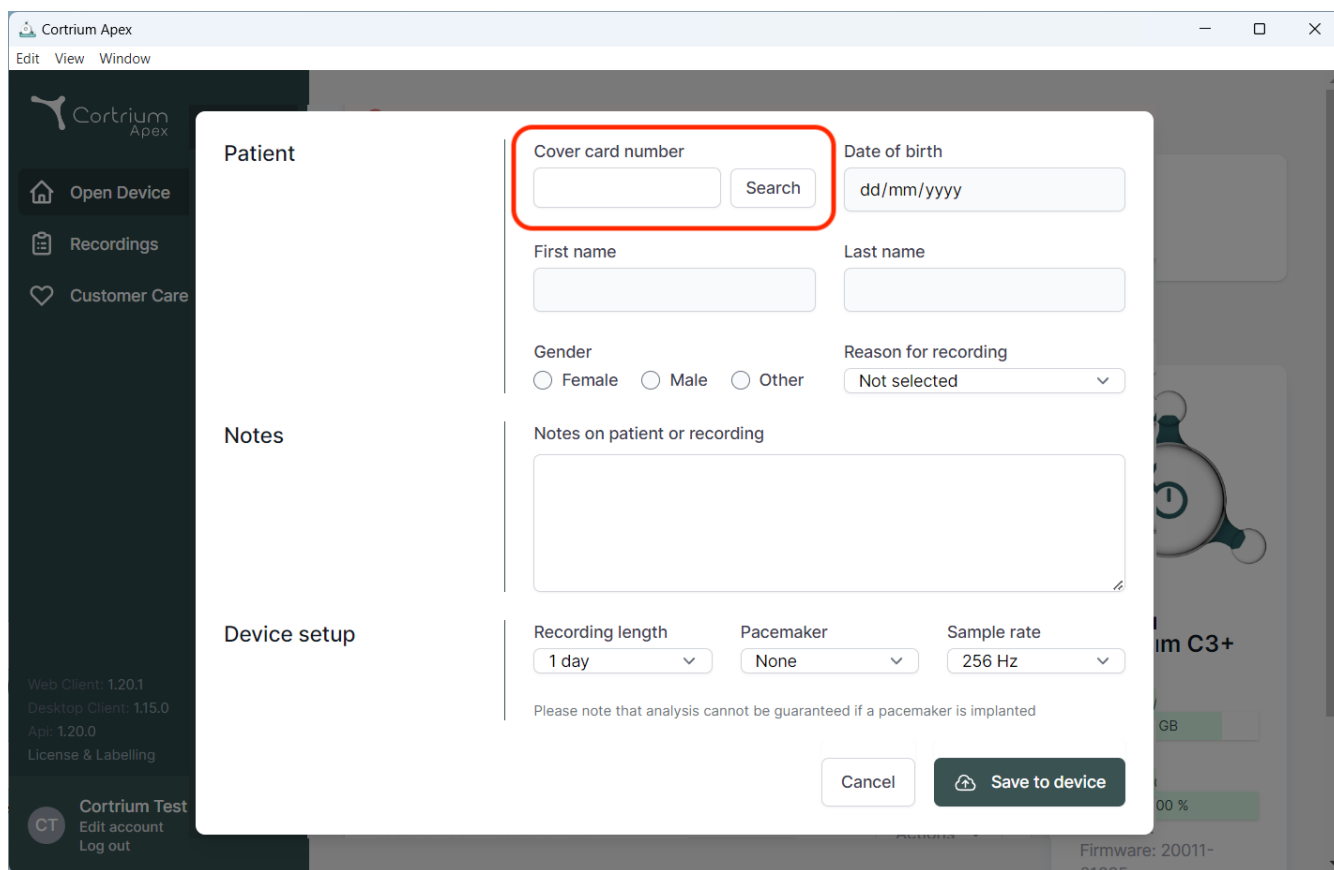
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient information. The form includes the following fields and controls:

- Cover card number:** A text input field, highlighted with a red box, with a "Search" button next to it.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes on patient or recording:** A large text area for notes.
- Device setup:** A section containing:
 - Recording length:** A dropdown menu showing "1 day".
 - Pacemaker:** A dropdown menu showing "None".
 - Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the form, there are two buttons: "Cancel" and "Save to device".