Cortrium Apex 1.26

Cortrium Apex 1.26

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
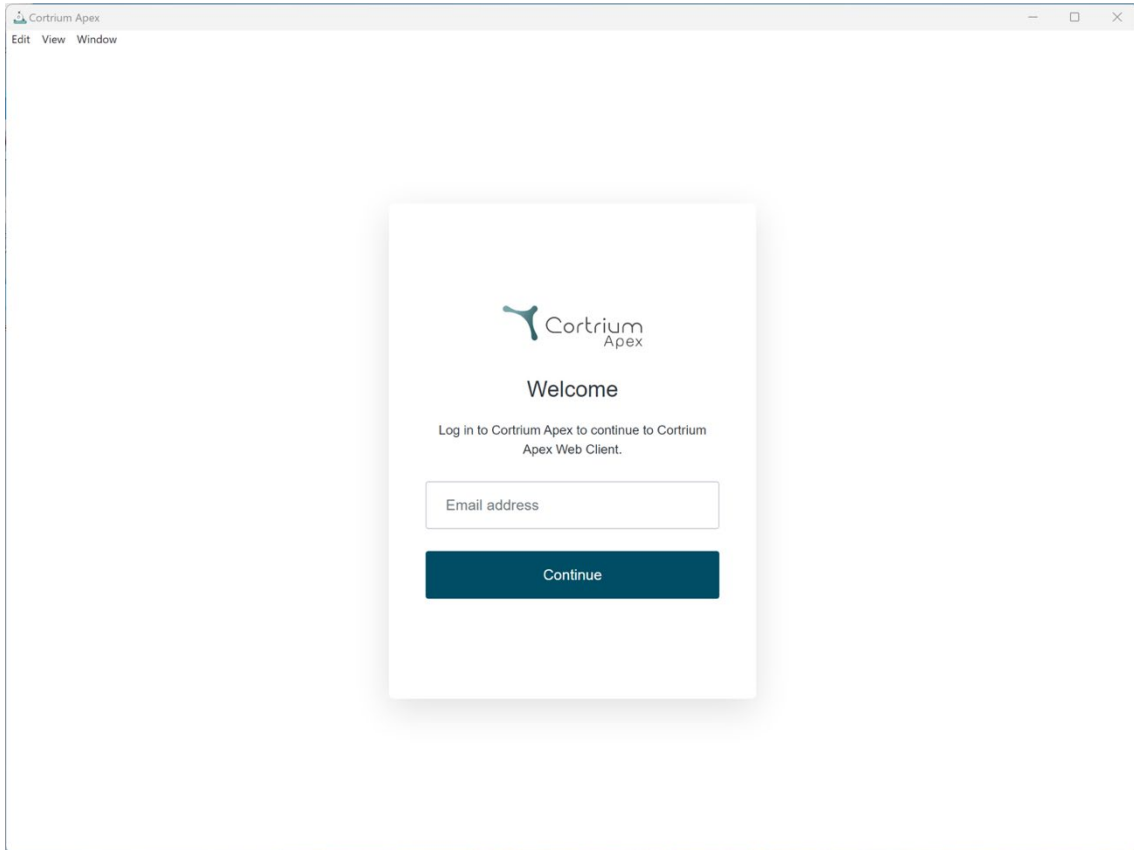
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". In the center of the browser is a white login dialog box with a shadow. The dialog box contains the Cortrium Apex logo, the word "Welcome", and the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." Below this is a text input field labeled "Email address" and a dark blue button labeled "Continue".

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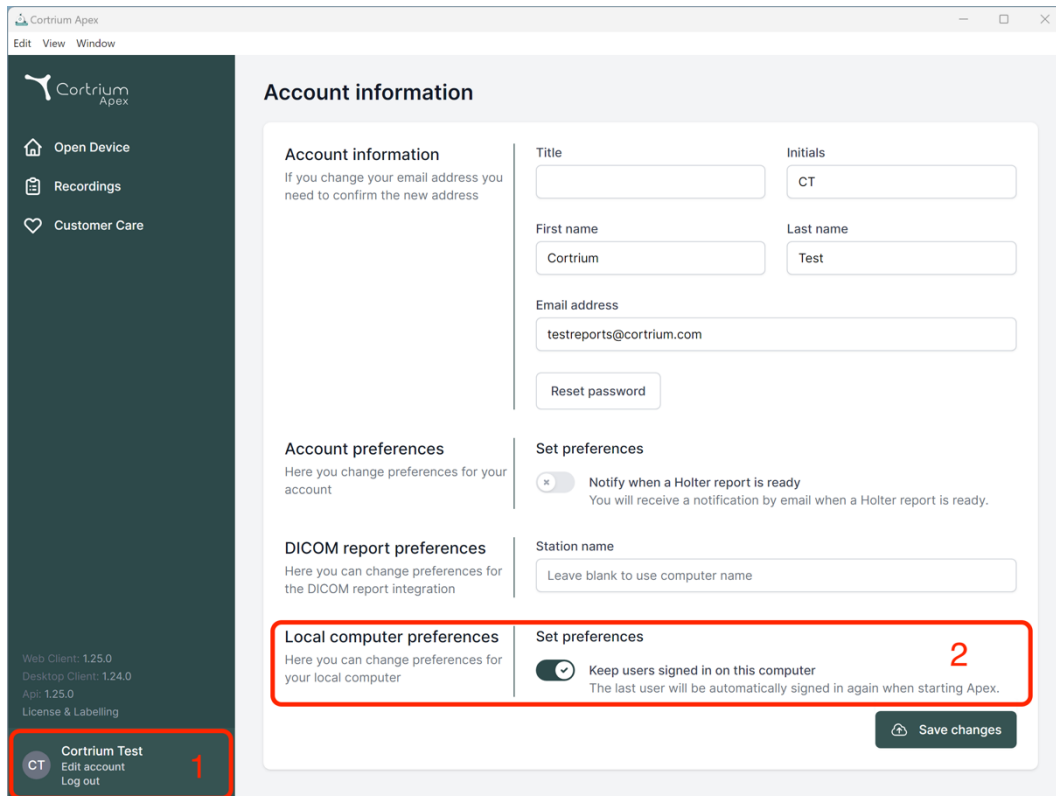☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

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Station name:

Local computer preferences

Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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
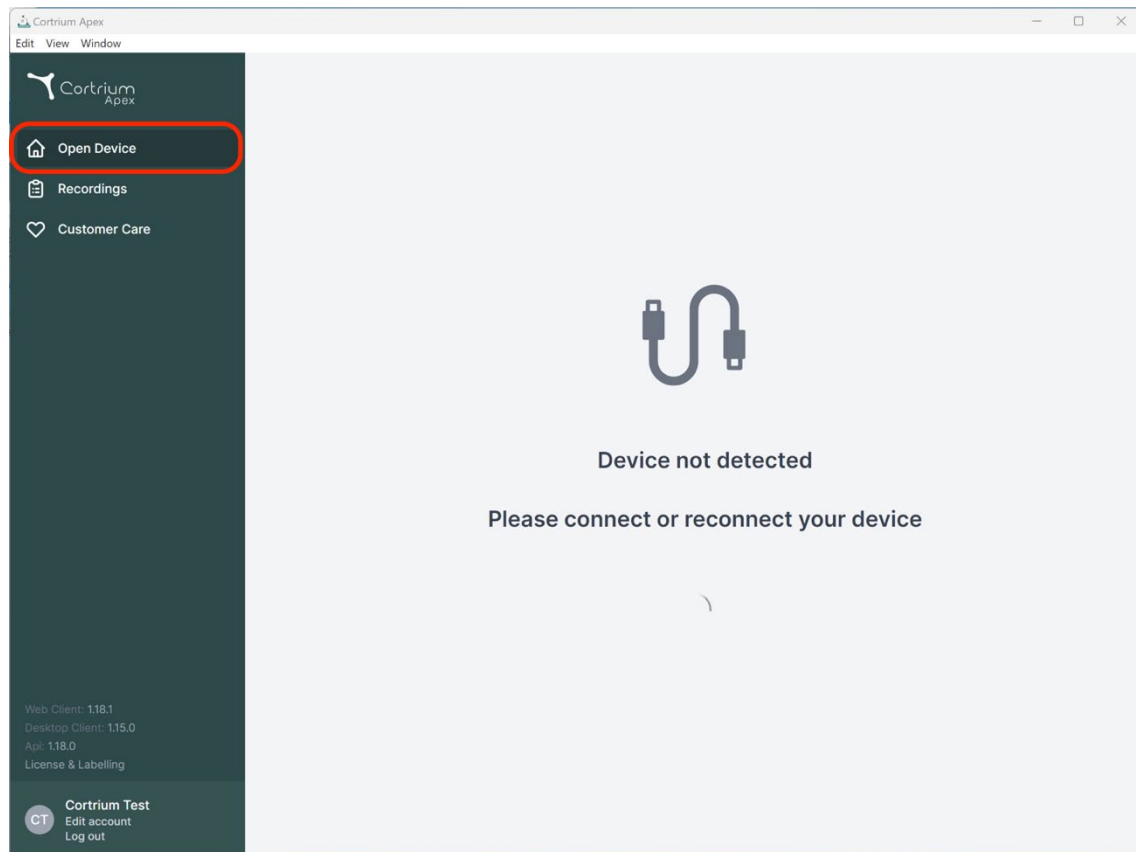
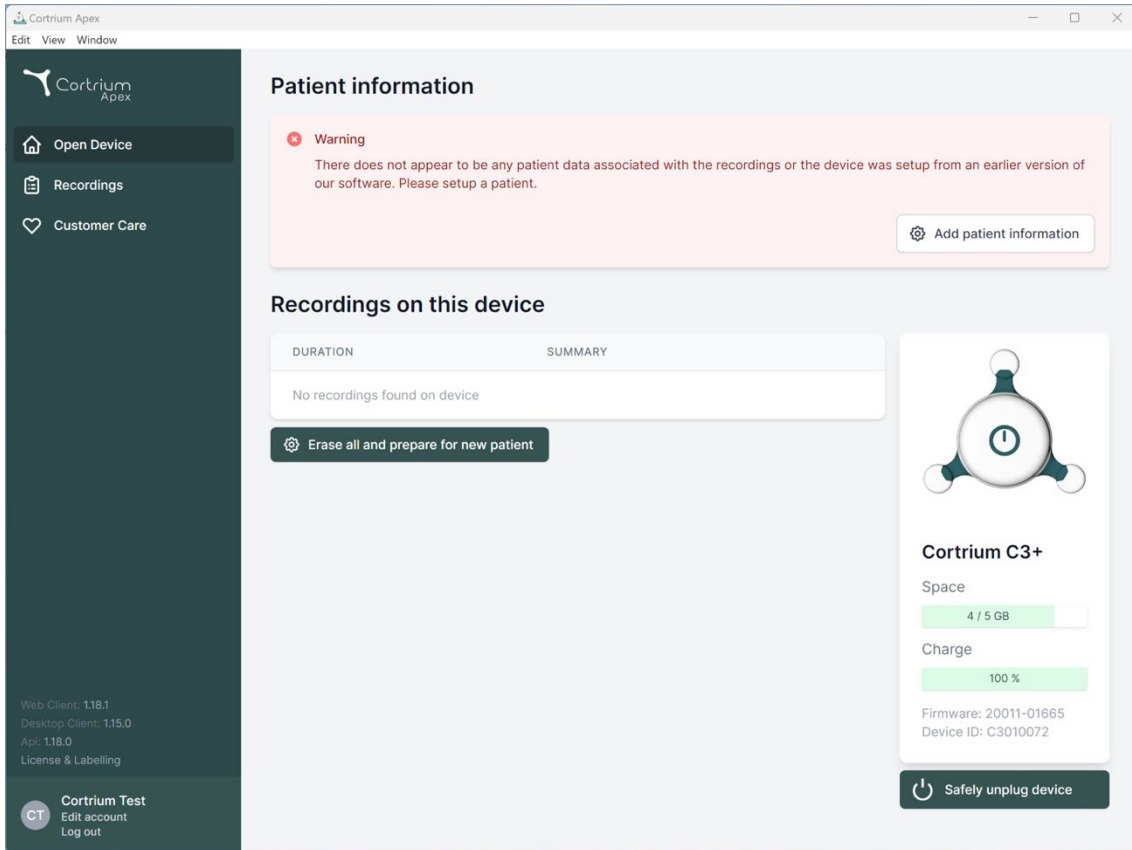
The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links.

The main content area has a light blue header with 'Patient information'. Below this is a pink warning box with a red 'x' icon and the text: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' A button 'Add patient information' is in the bottom right of the warning box.

Below the warning box is a section 'Recordings on this device'. It contains a table with columns 'DURATION' and 'SUMMARY'. The table is empty, showing 'No recordings found on device'. Below the table is a button 'Erase all and prepare for new patient'.

On the right side of the main content area is a device status card for 'Cortrium C3+'. It features a circular icon with a clock. Below the icon, it shows 'Space' usage as '4 / 5 GB' with a green progress bar, and 'Charge' status as '100 %' with a green progress bar. At the bottom of the card, it lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A button 'Safely unplug device' is at the bottom of the card.

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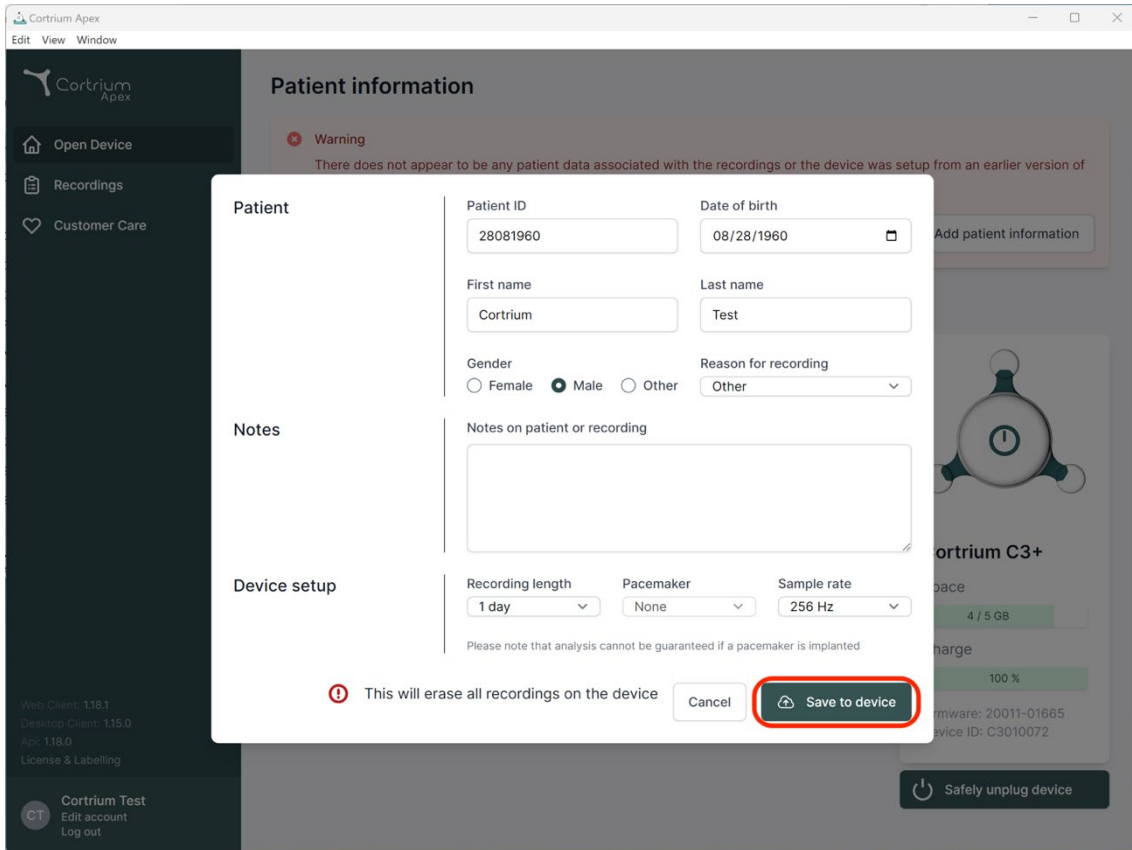Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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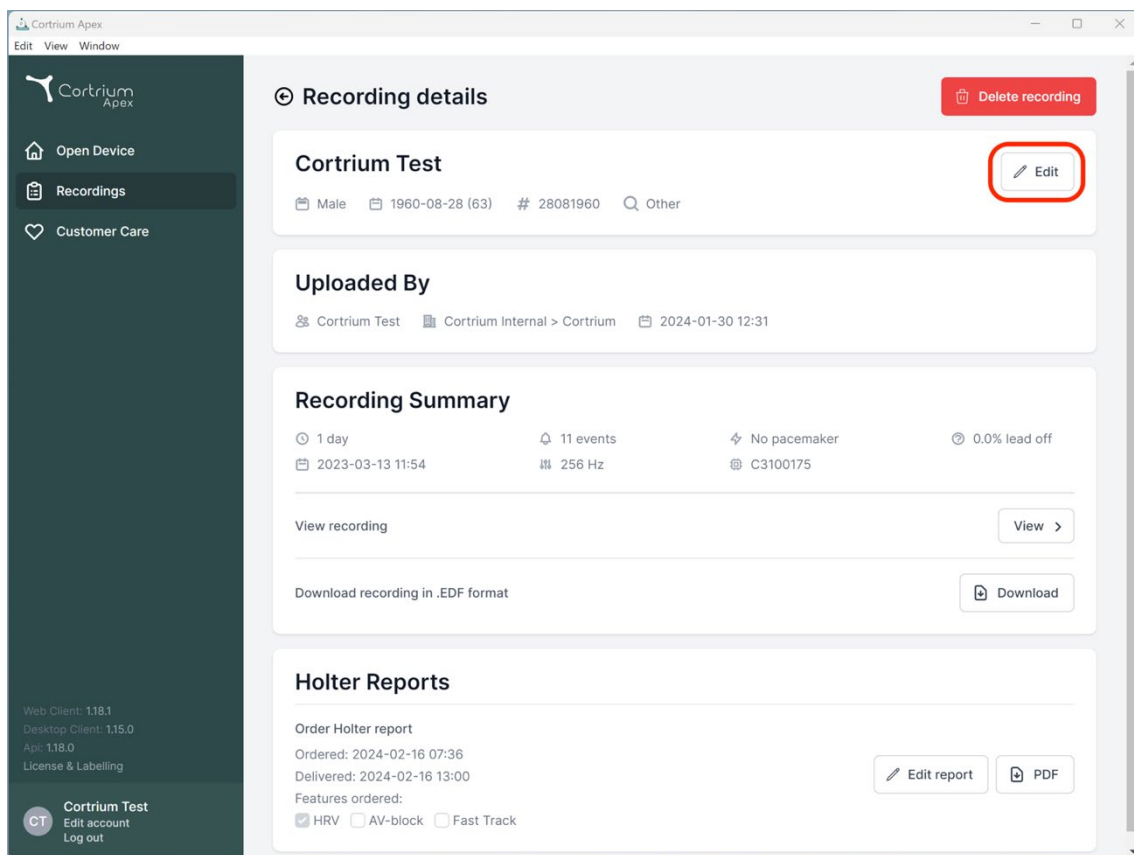
2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
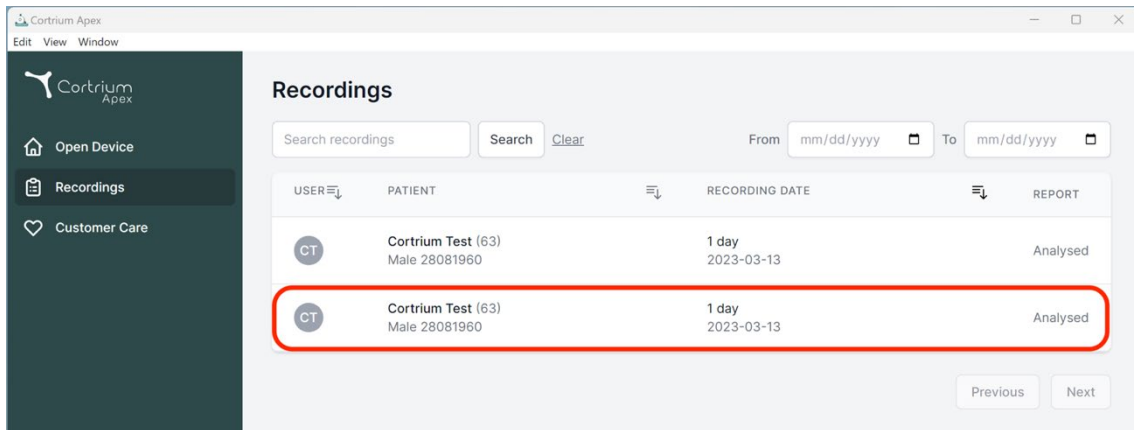
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

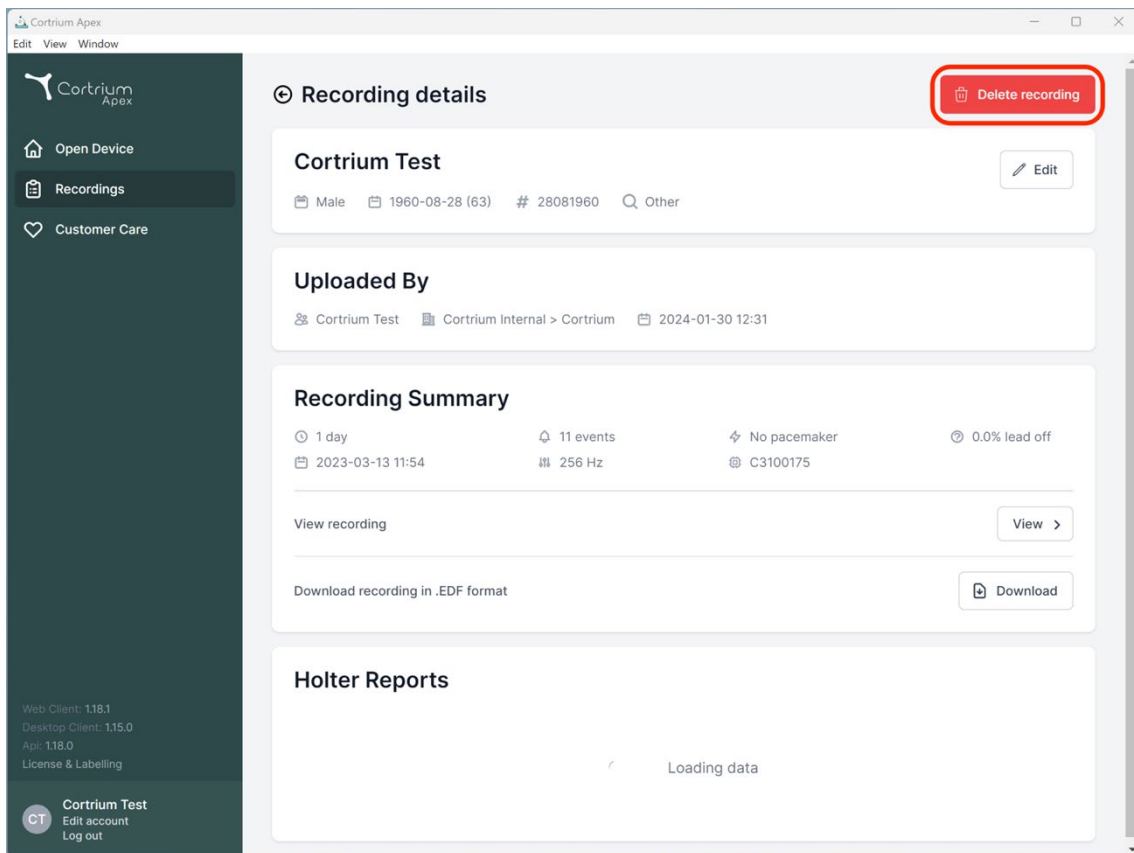Previous Next

Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View](#)

Download recording in .EDF format [Download](#)

Holter Reports

Loading data

Web Client: 1.18.1
Desktop Client: 1.15.0
API: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

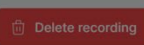
Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Recording details 

Cortrium Test 

 Male  1960-08-28 (63)  # 28081960  Other

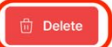
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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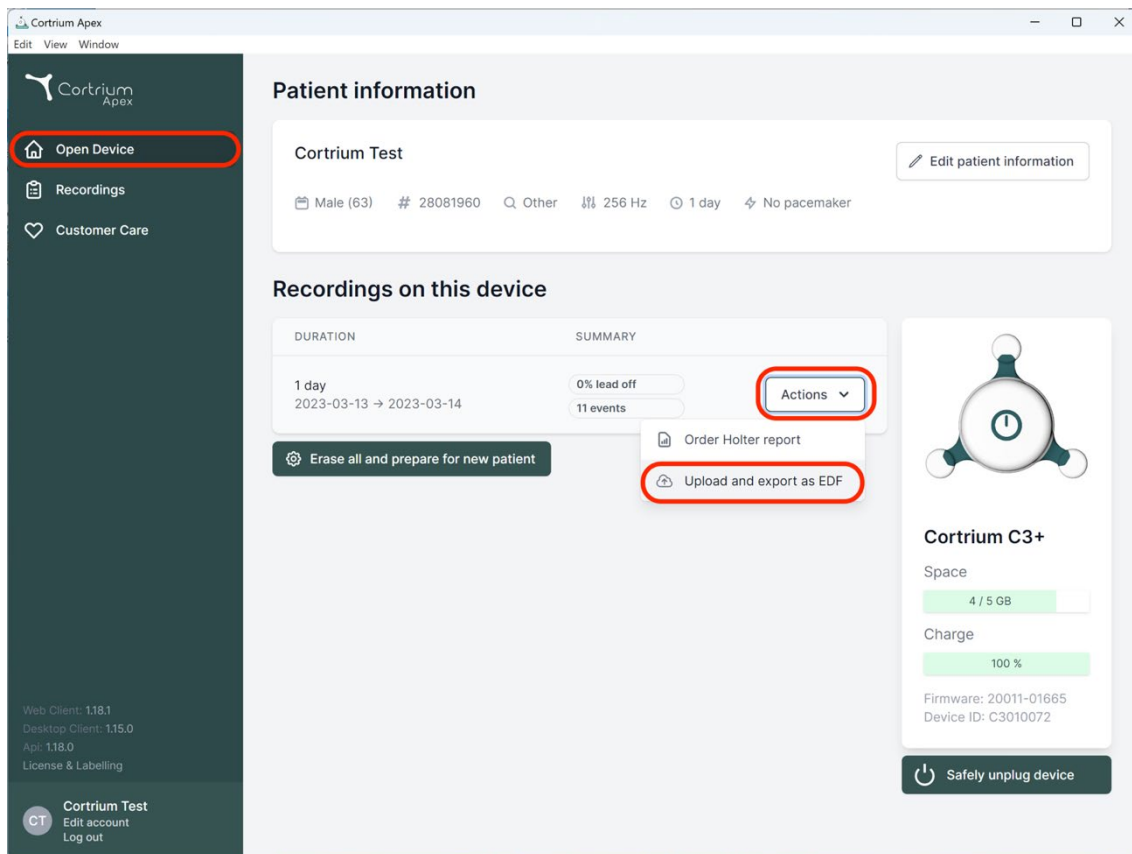
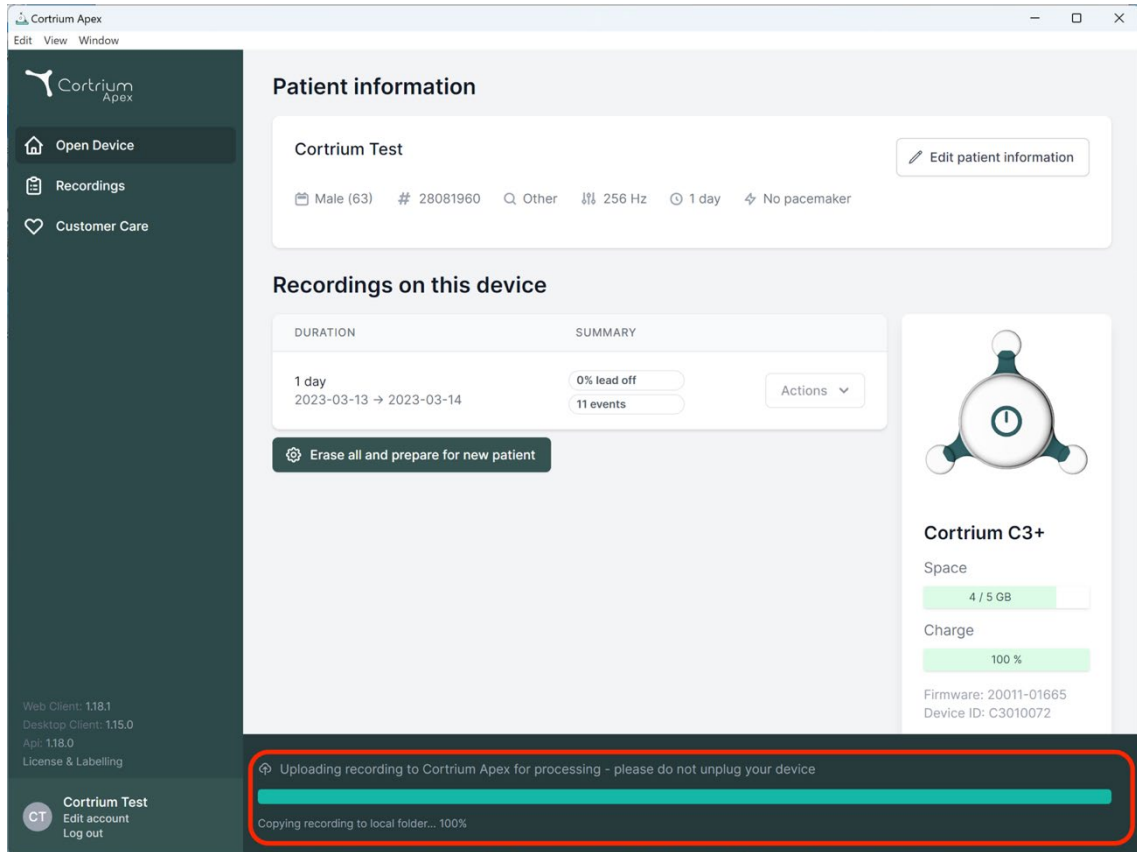
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

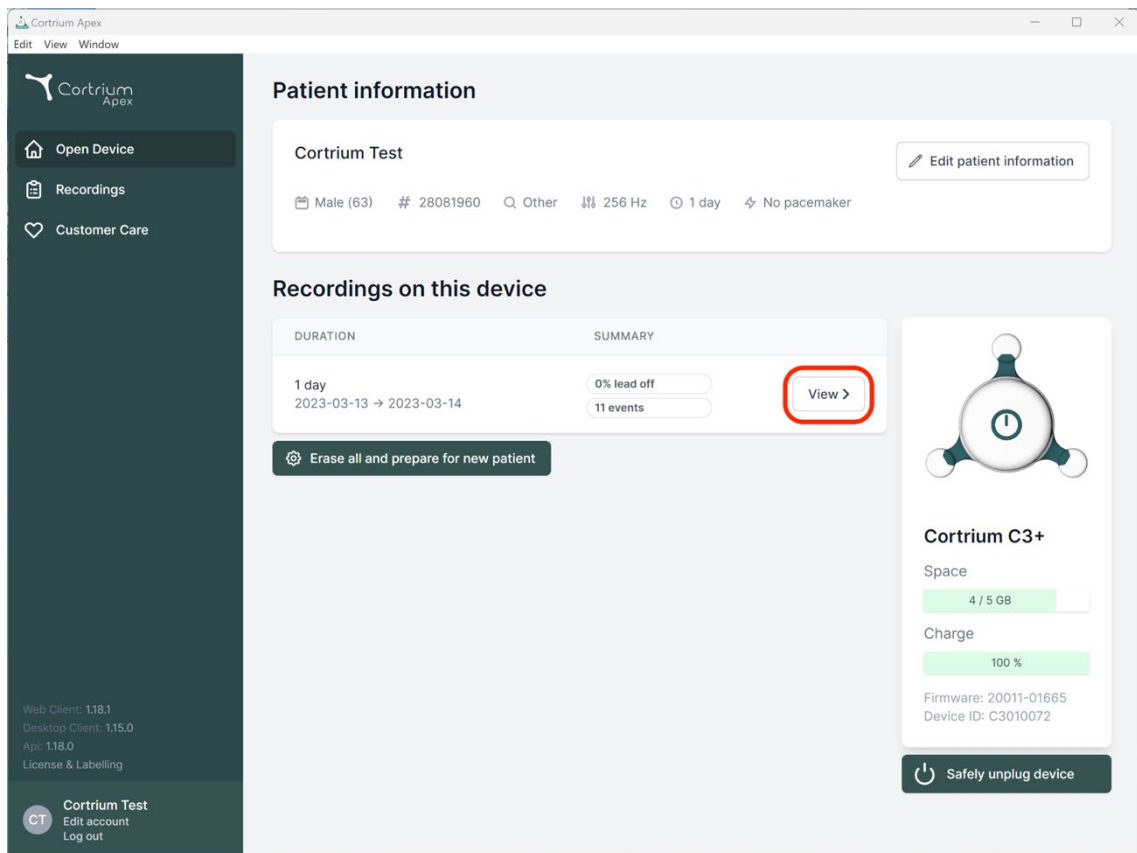
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
CT Edit account Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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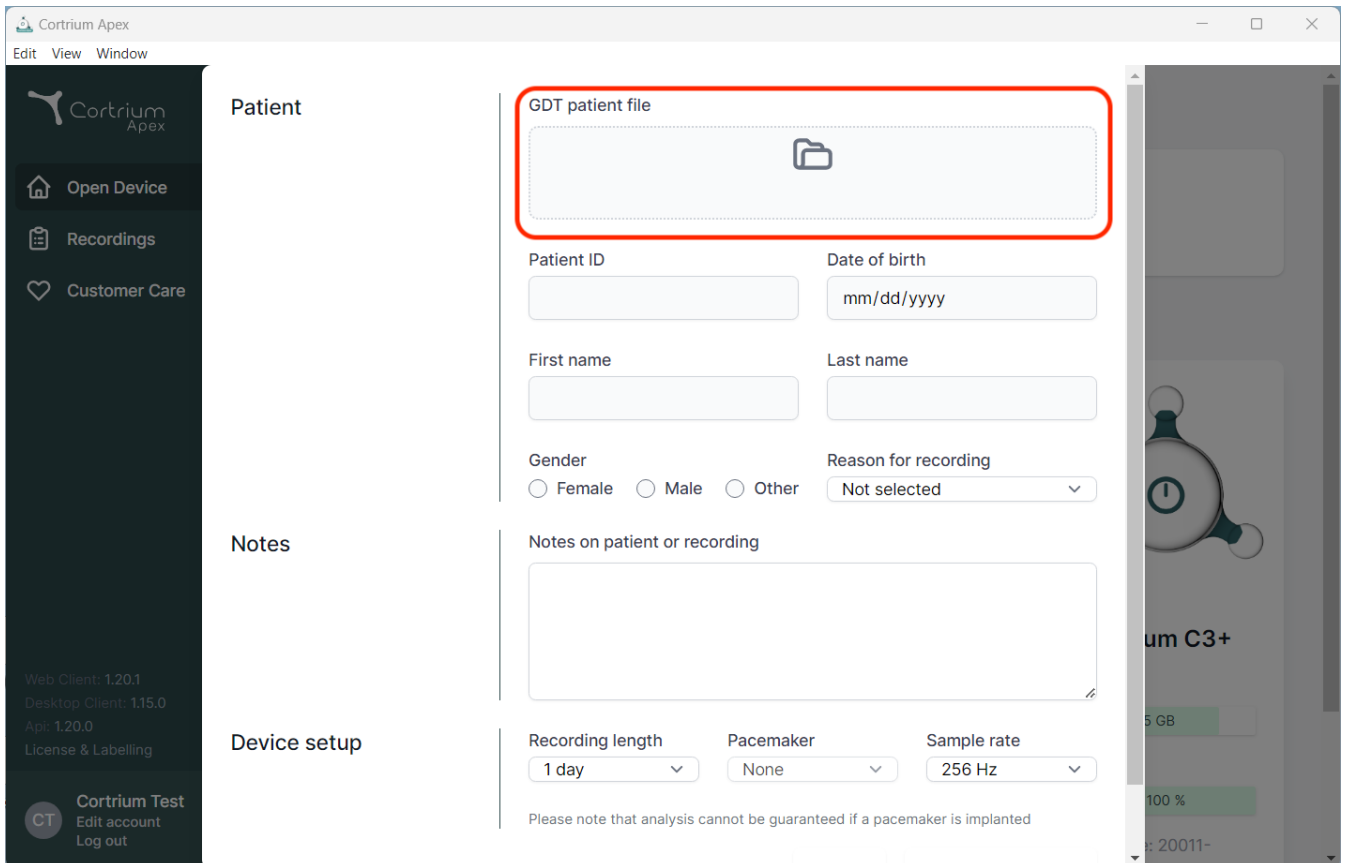
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

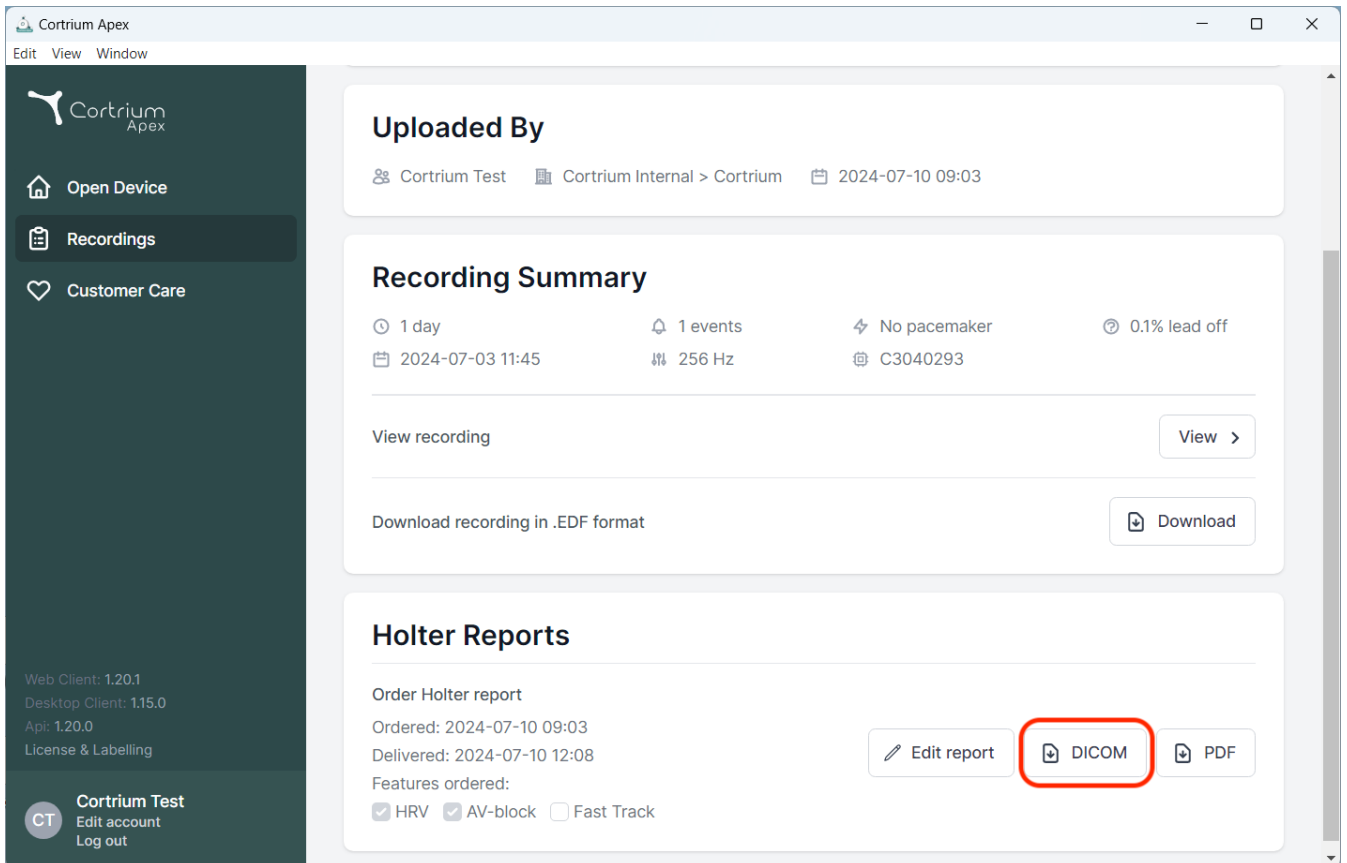
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section has a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below these fields states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device model, "um C3+", is visible, showing a battery level of "100 %" and a date "20011-".

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling) and a user profile for 'Cortrium Test'. The main content area has a light gray background. The top section, 'Uploaded By', shows 'Cortrium Test' and a date of '2024-07-10 09:03'. Below this is the 'Recording Summary' section, which includes a 'View recording' button and a 'Download recording in .EDF format' button. The bottom section, 'Holter Reports', shows the 'Order Holter report' details, including the order date (2024-07-10 09:03) and delivery date (2024-07-10 12:08). It also lists ordered features: HRV, AV-block, and Fast Track. At the bottom right of this section are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

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Account information

Account information

If you change your email address you need to confirm the new address

Title

Initials

CT

First name

Cortrium

Last name

Test

Email address

testreports@cortrium.onmicrosoft.com

Reset password

Account preferences

Here you change preferences for your account

Set preferences

×

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

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
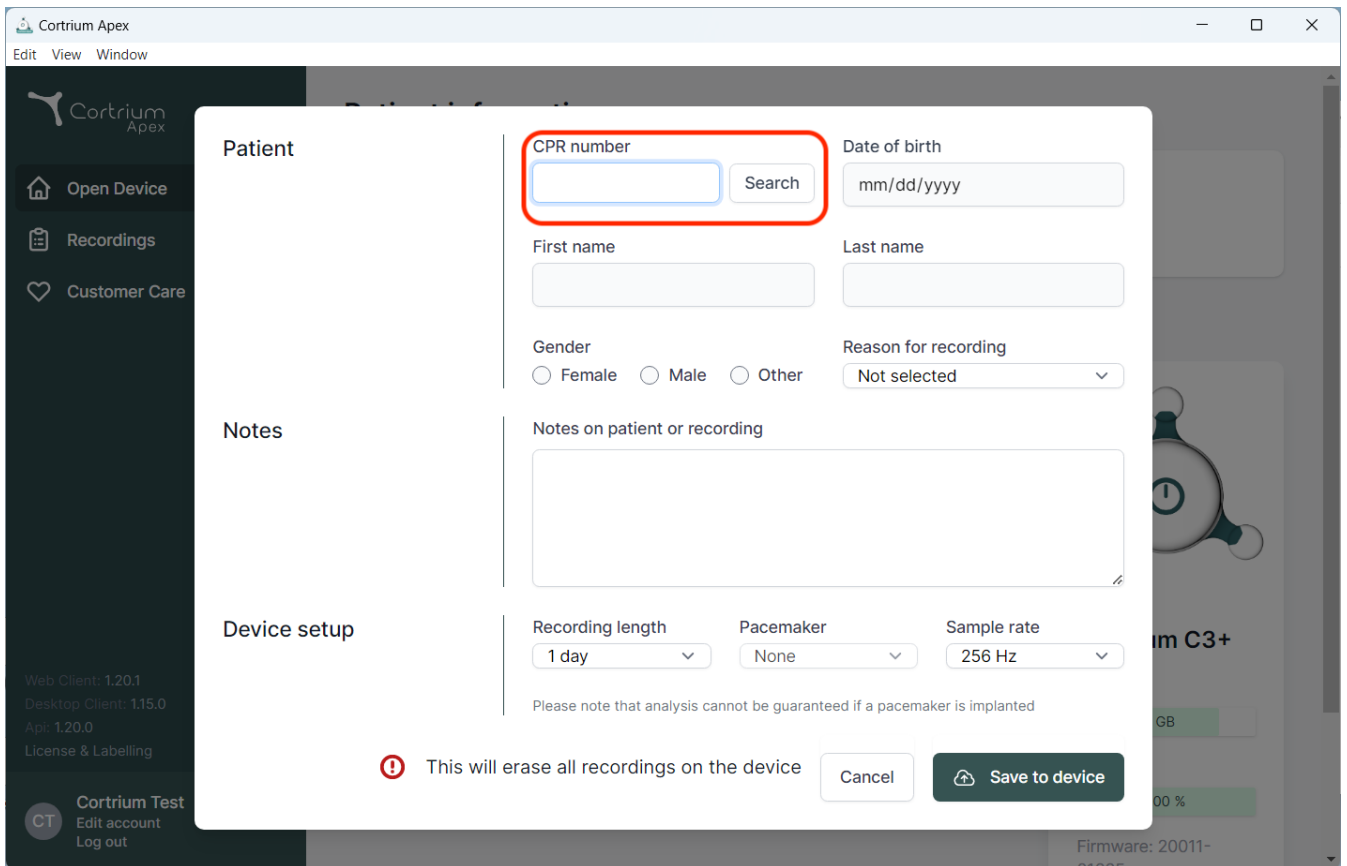
The screenshot shows the Cortrium Apex software interface with a modal window for patient search. The modal is titled "Patient" and contains the following fields and controls:

- CPR number:** A text input field with a red border, followed by a "Search" button.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.
- Notes:** A section titled "Notes" with a text area labeled "Notes on patient or recording".
- Device setup:** A section with three dropdown menus: "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, there is a warning message: "⚠ This will erase all recordings on the device". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

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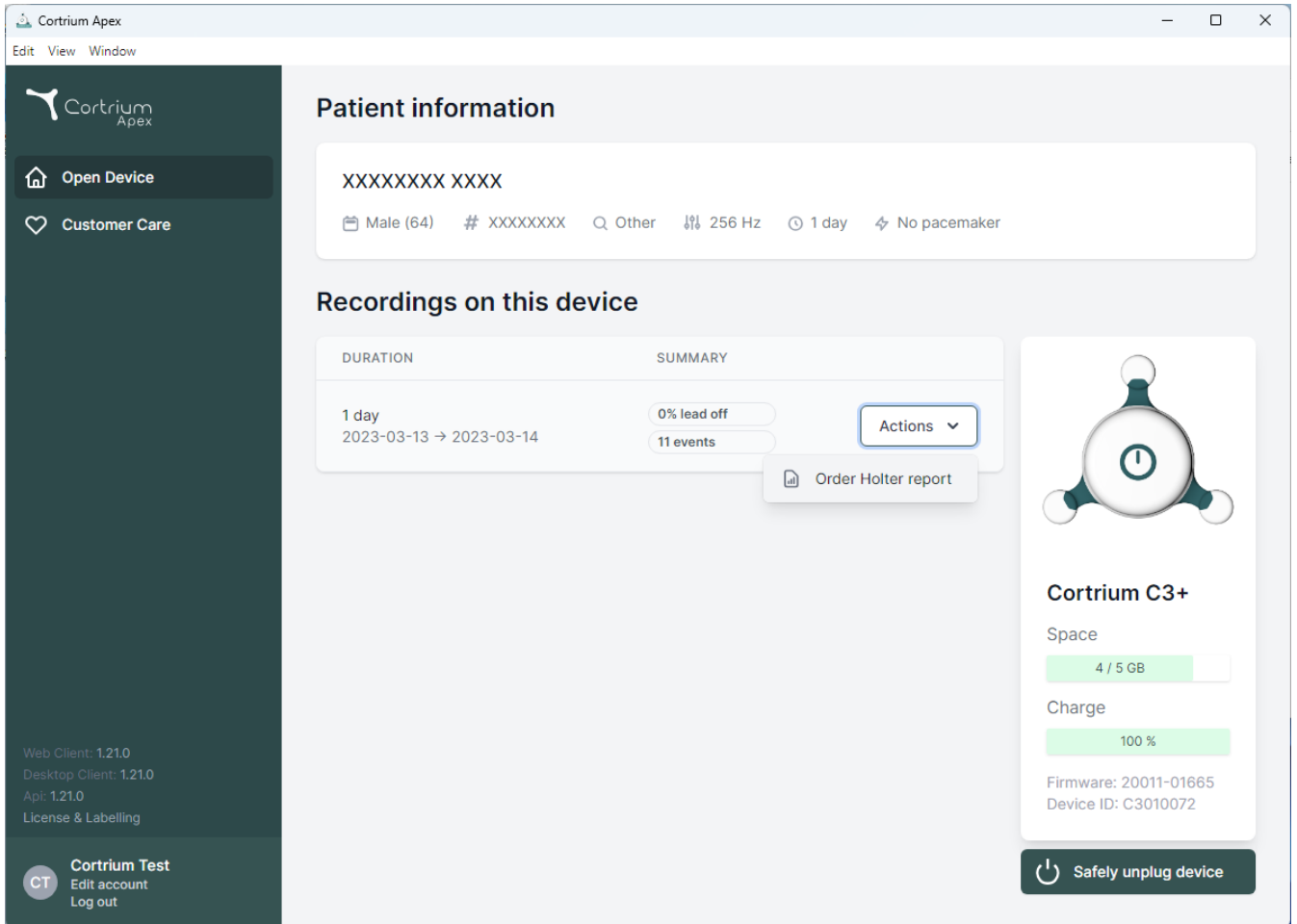The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

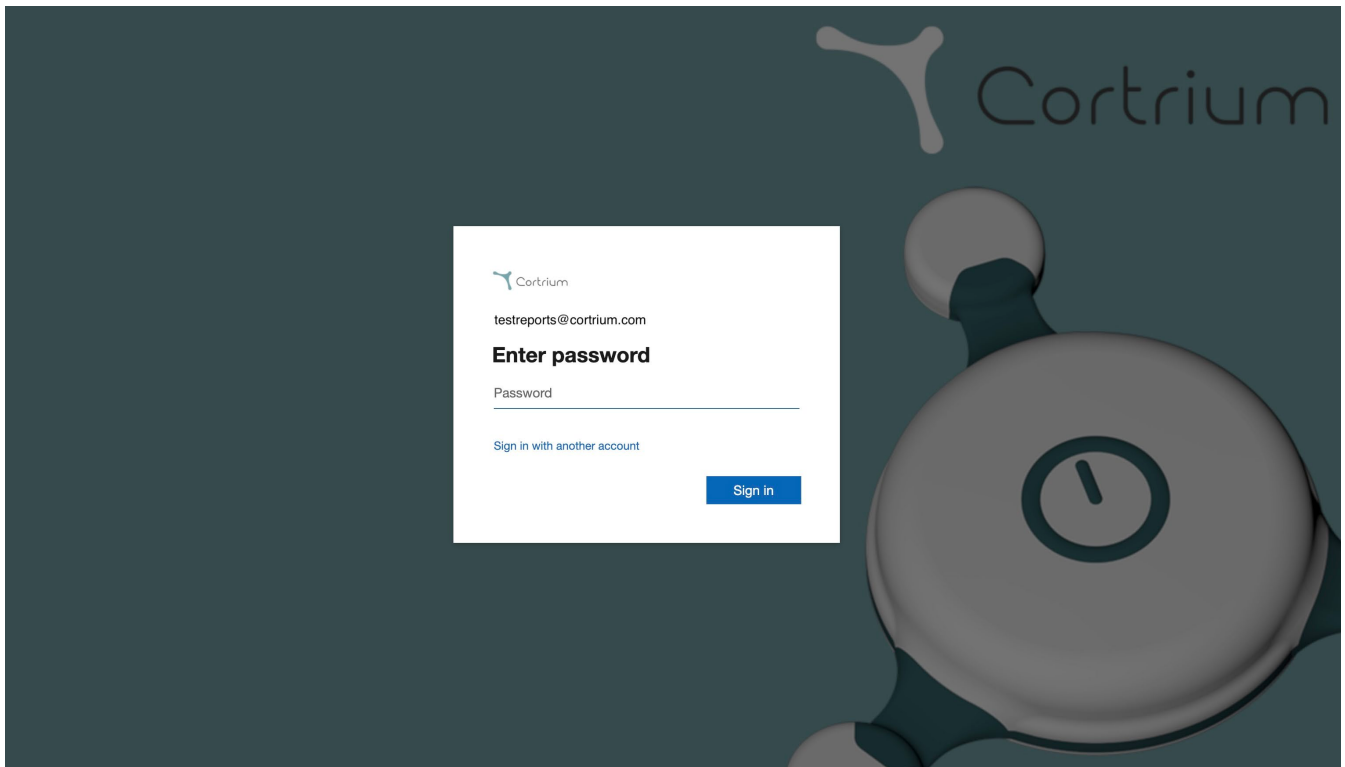
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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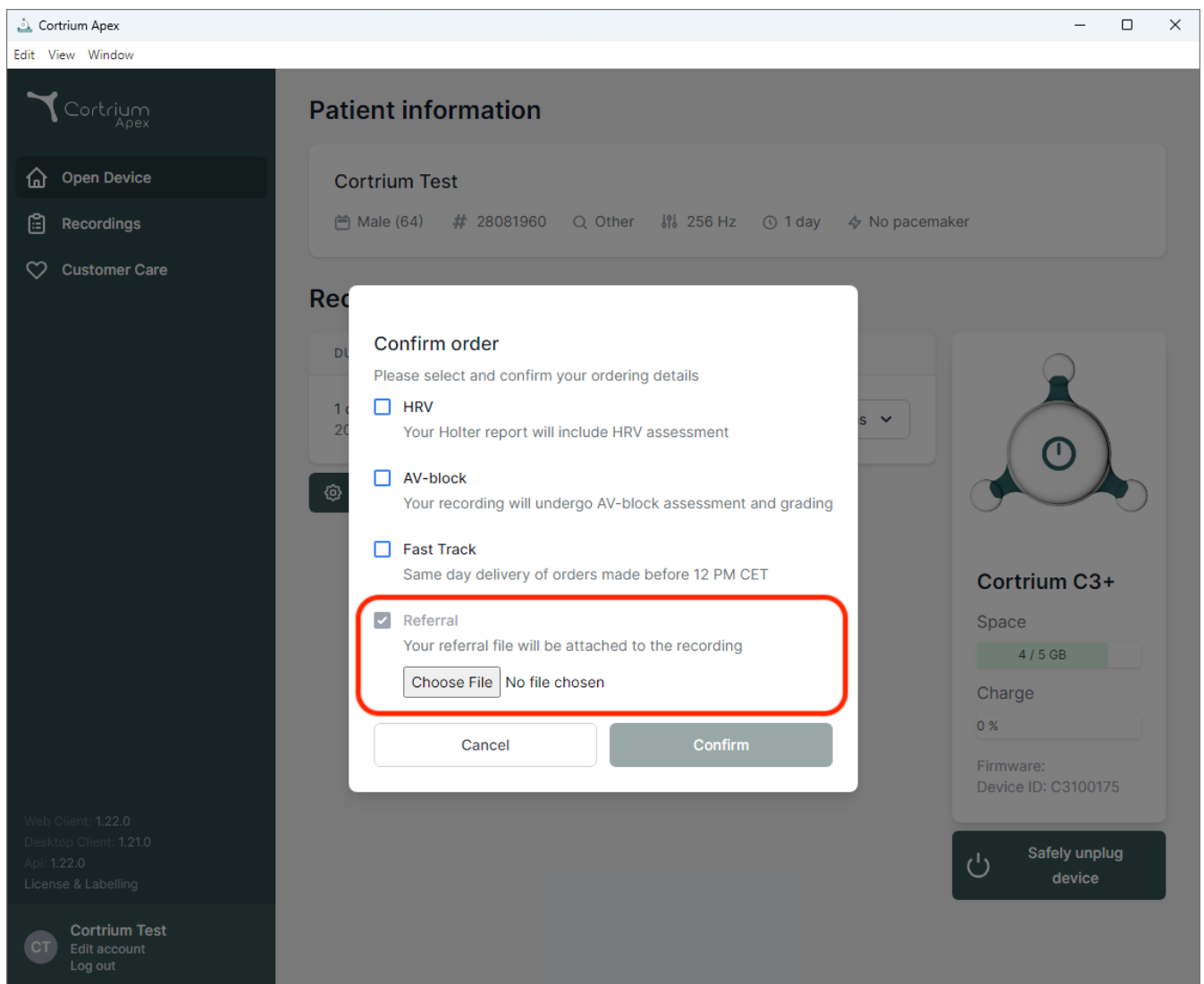
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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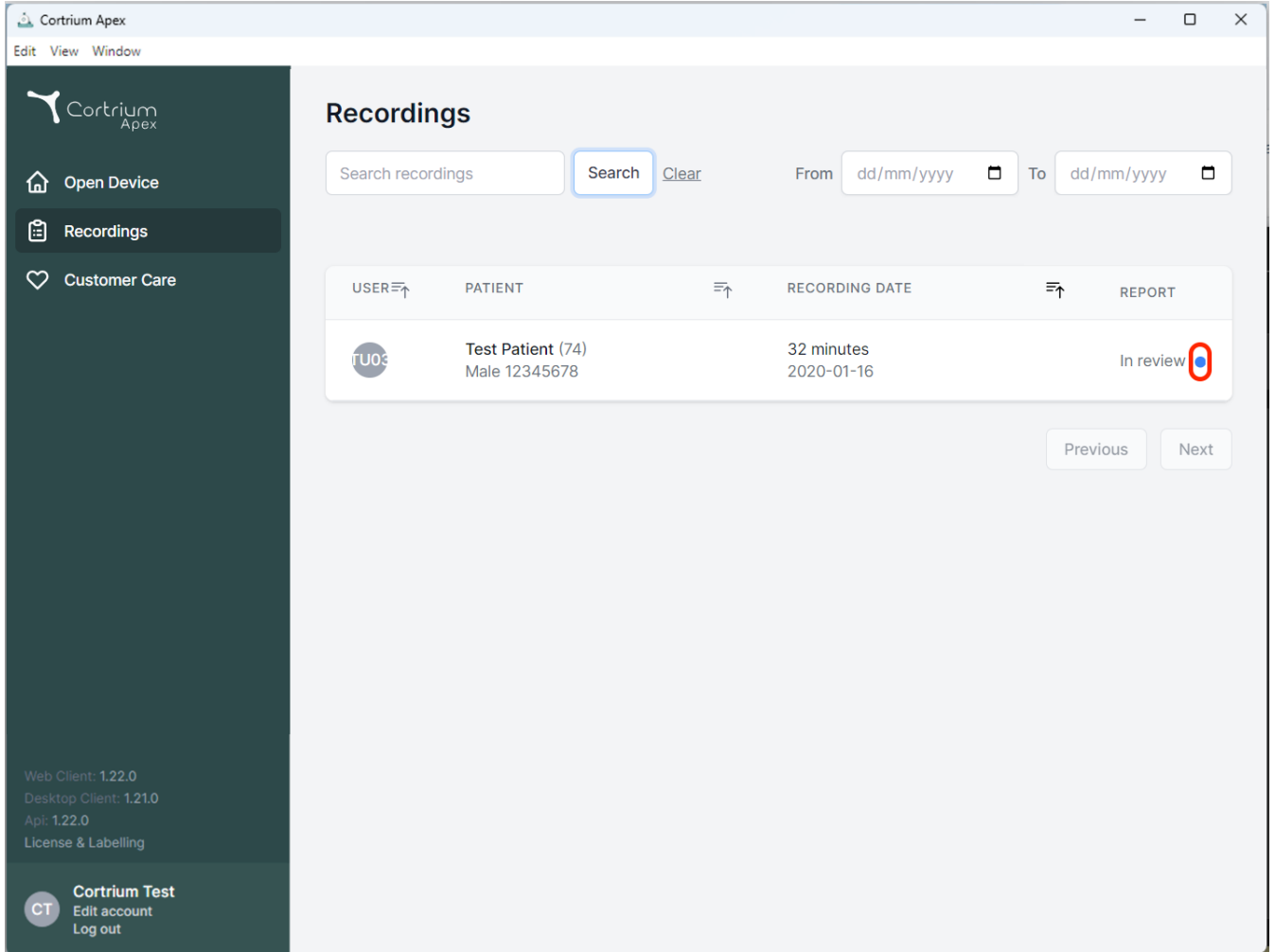
Recordings

Search recordings [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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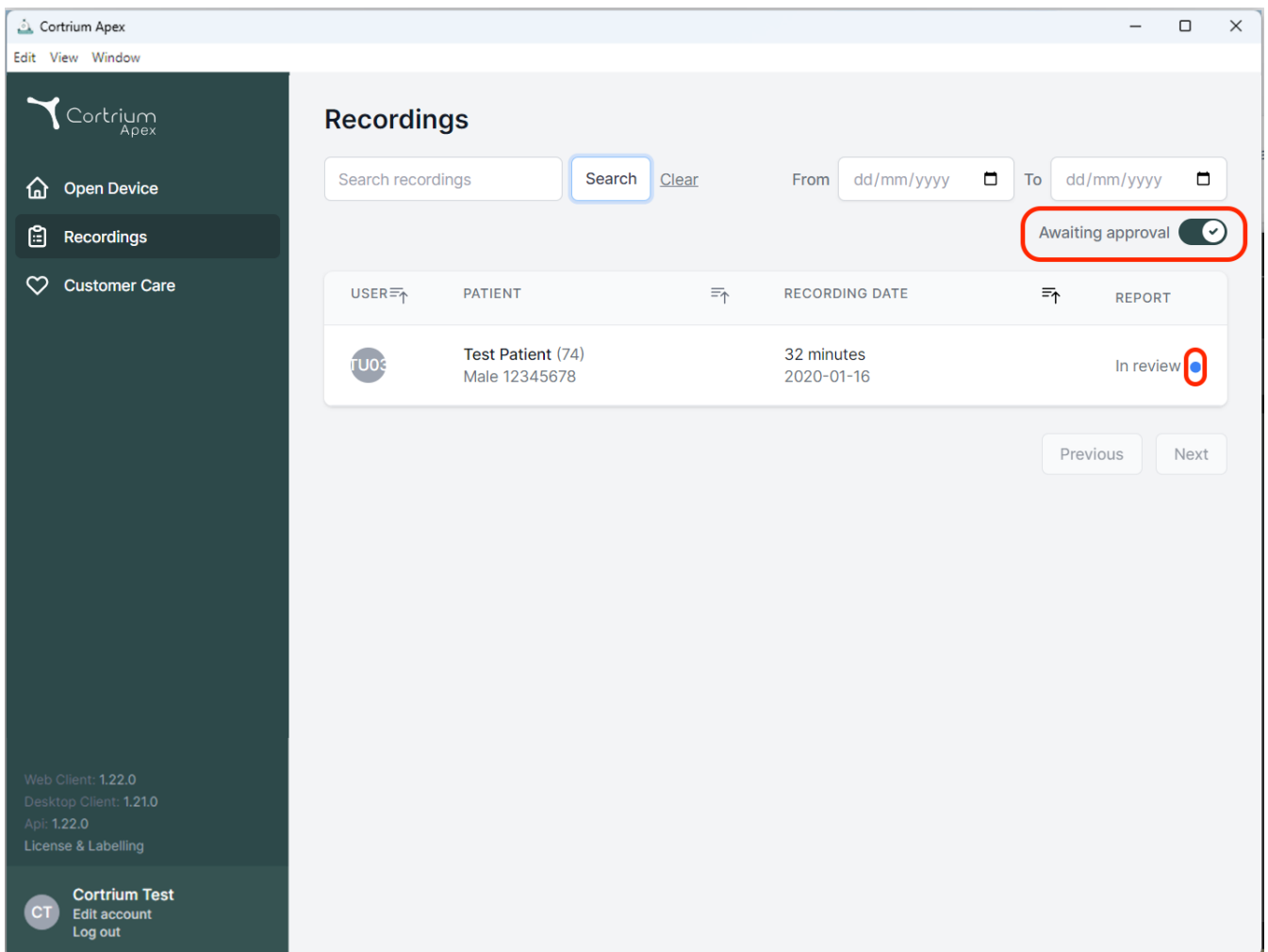
Search recordings From To


Awaiting approval ☒

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●

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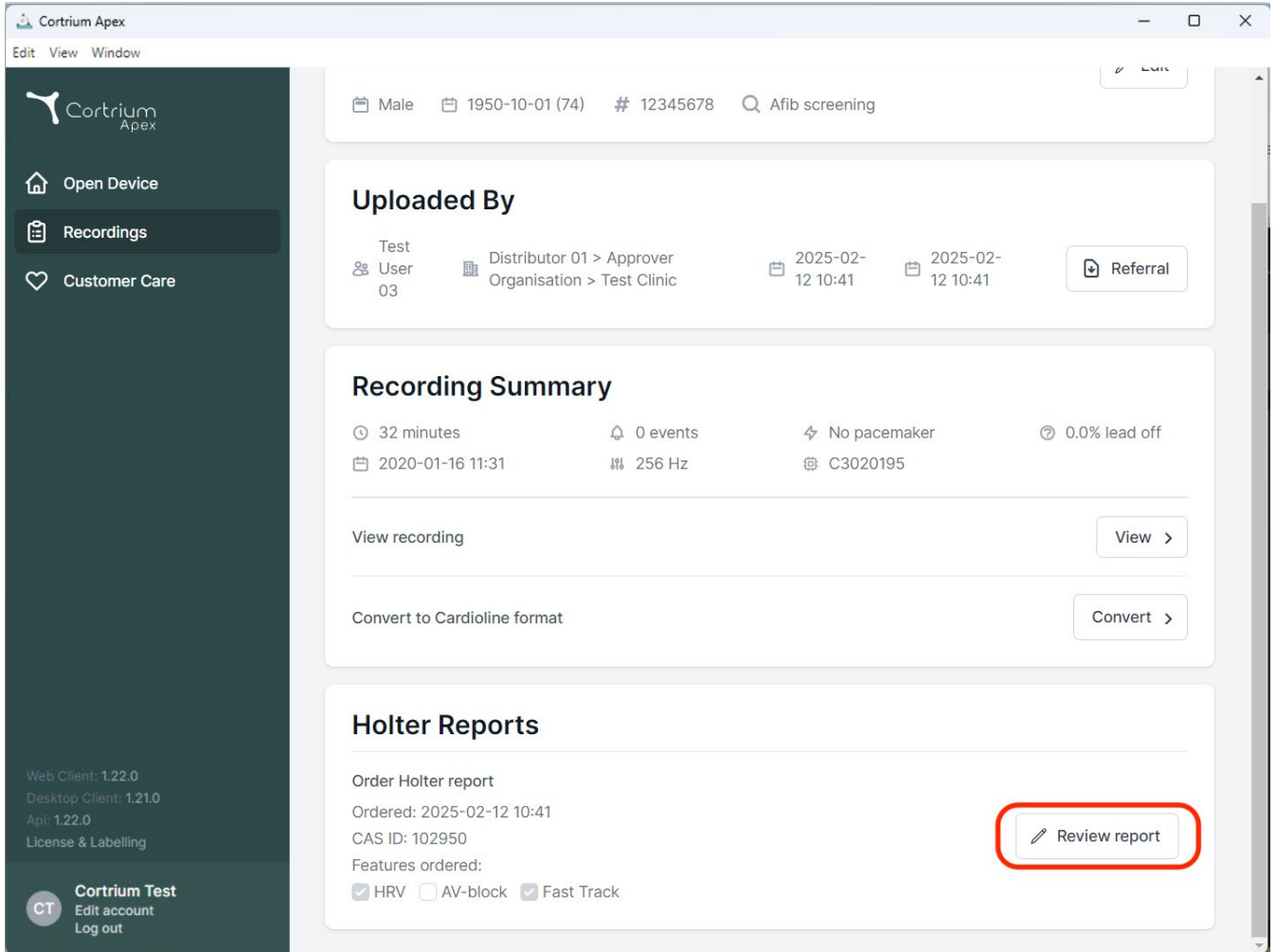
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41 and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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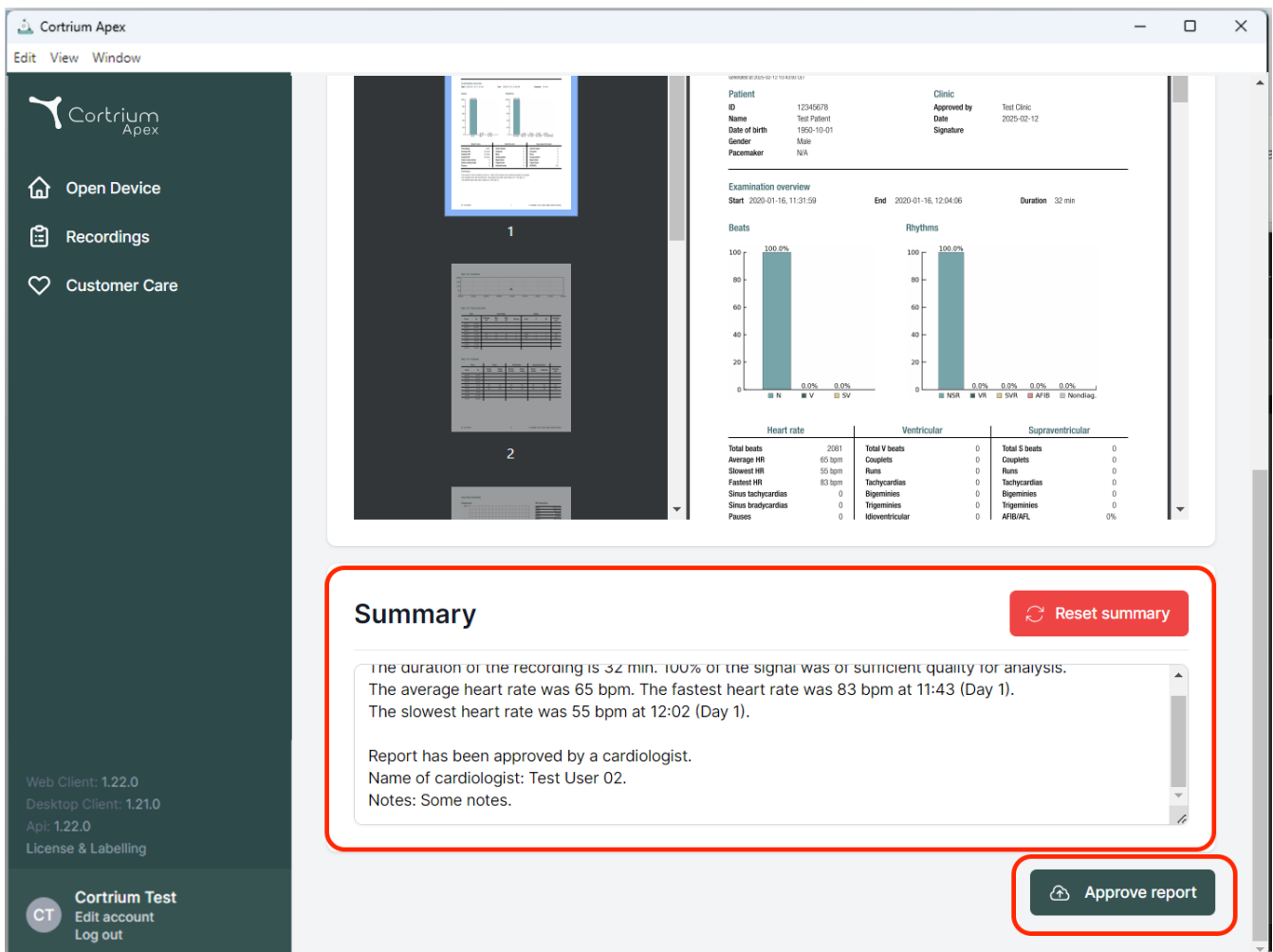Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Apl: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Examination overview
Start: 2020-01-16, 11:31:59 | End: 2020-01-16, 12:04:06 | Duration: 32 min

Beats
100.0%
0.0%
0.0%

Rhythms
100.0%
0.0%
0.0%
0.0%
0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2031	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigemines	0	Bigemines	0
Sinus bradycardias	0	Trigemines	0	Trigemines	0
Pauses	0	Idioventricular	0	AF/IB/AFI	0%

Summary [Reset summary](#)

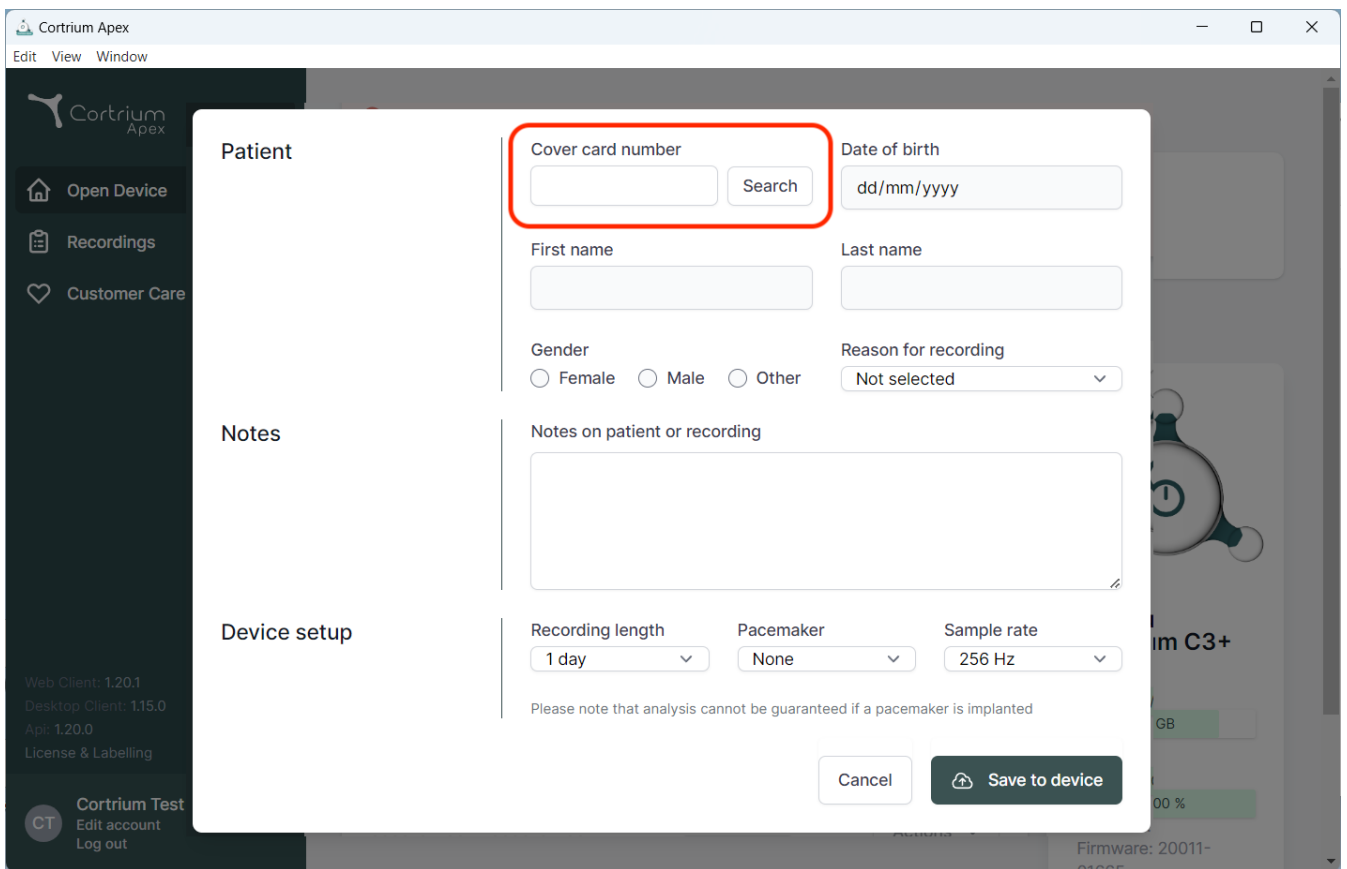
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled 'Patient' is open, showing a search form. The 'Cover card number' field is highlighted with a red box, and a 'Search' button is next to it. Other fields include 'Date of birth' (dd/mm/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), and 'Reason for recording' (Not selected). Below these is a 'Notes on patient or recording' text area. The 'Device setup' section includes 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A disclaimer states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. At the bottom are 'Cancel' and 'Save to device' buttons.

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denemarken
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
46 / 46