Cortrium Apex 1.26

Cortrium Apex 1.26

Bruksanvisning

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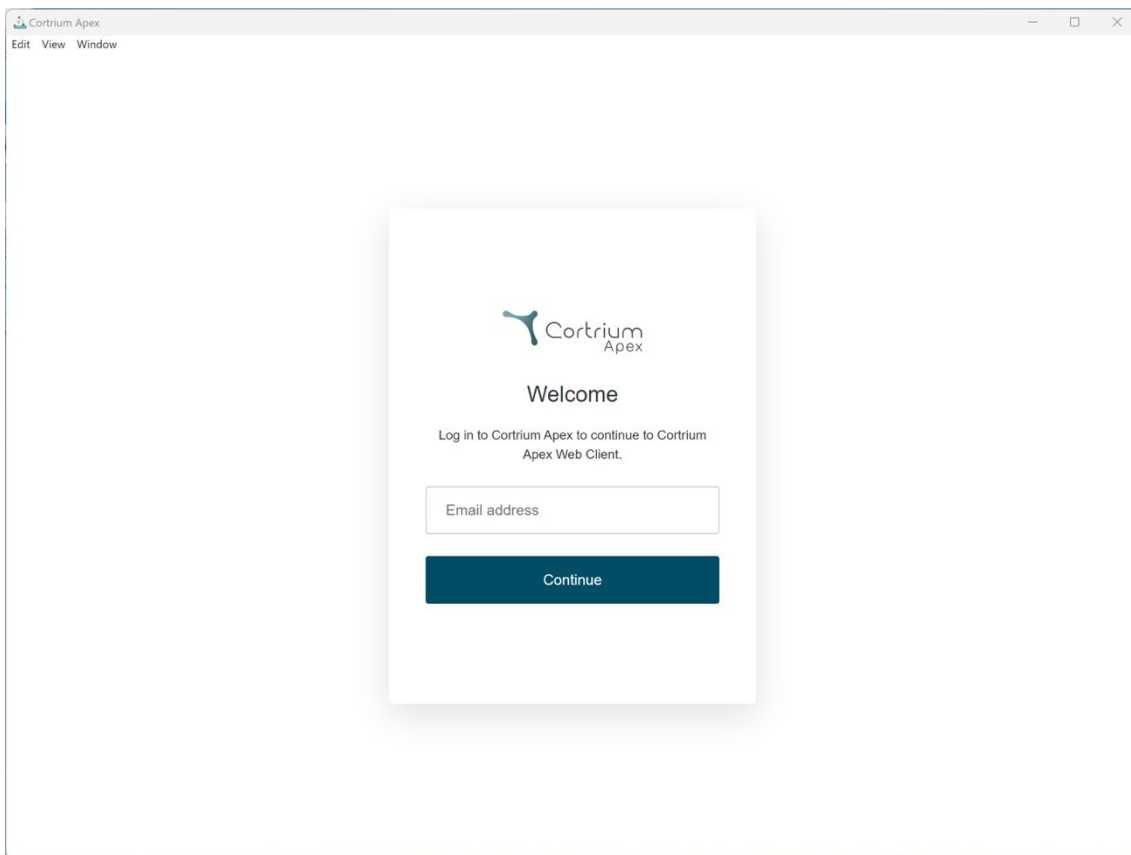
 Cortrium
Apex

Welcome

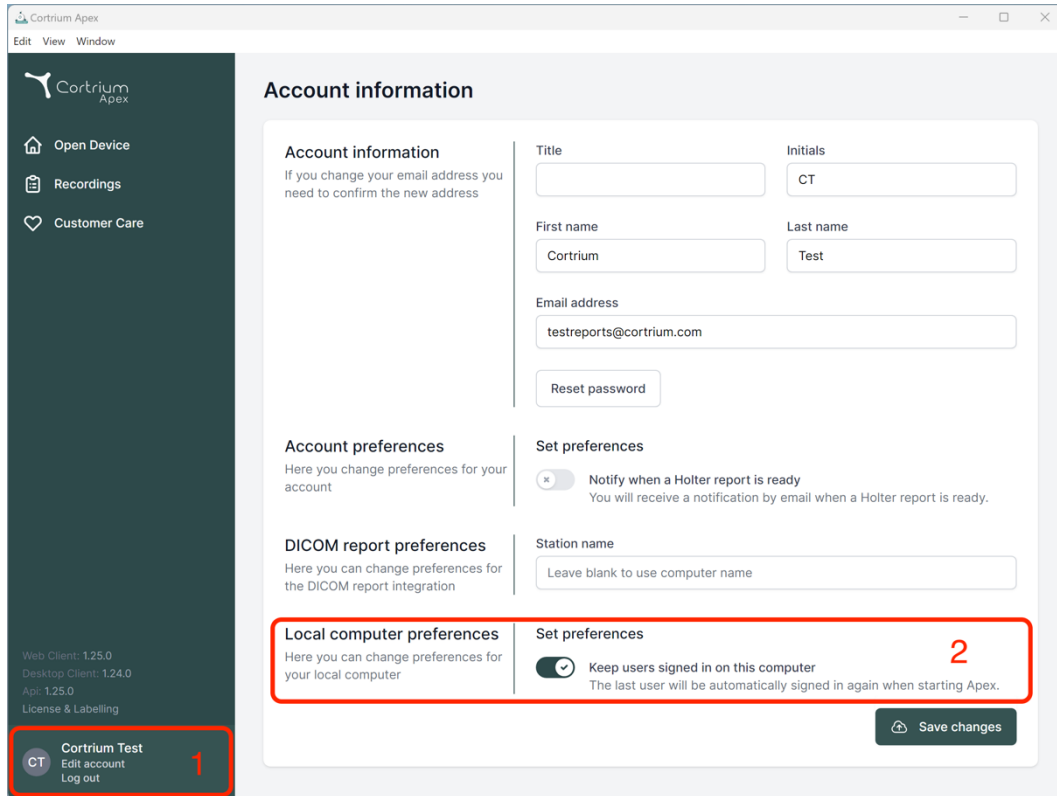
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0) and a user profile section for 'Cortrium Test' with 'Edit account' and 'Log out' options, highlighted by a red box and labeled '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address), 'Account preferences', 'DICOM report preferences', 'Set preferences' (with a toggle for 'Notify when a Holter report is ready'), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer'). The 'Local computer preferences' section is highlighted by a red box and labeled '2'. A 'Save changes' button is located at the bottom right of the main content area.

Account information

Account information
If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

DICOM report preferences
Here you can change preferences for the DICOM report integration

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

Station name

Local computer preferences
Here you can change preferences for your local computer

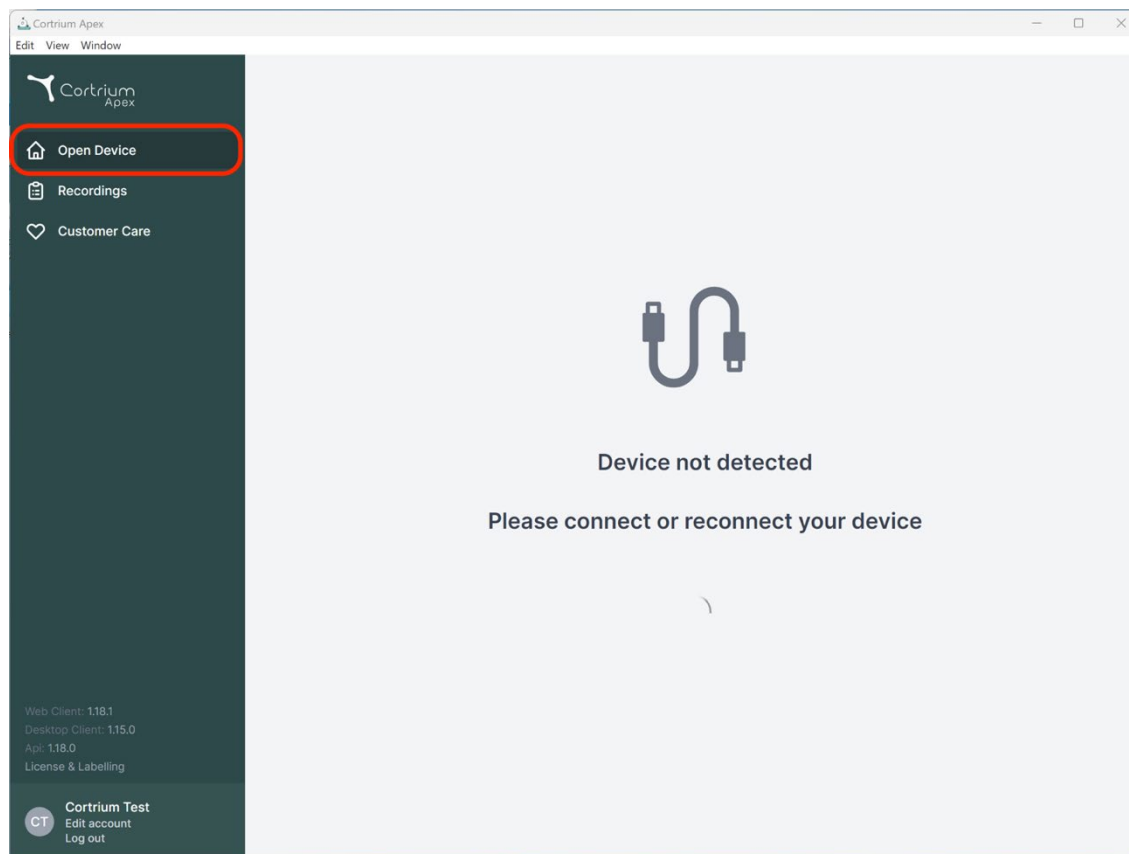
☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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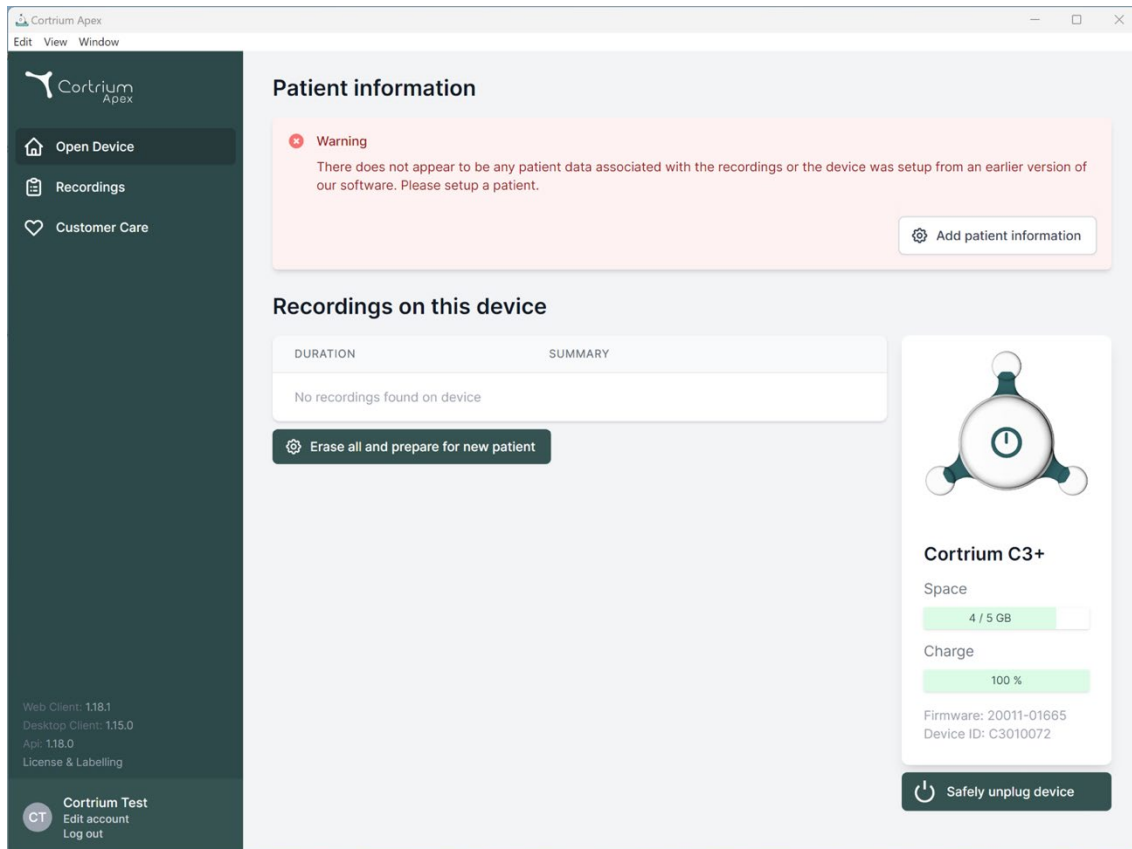
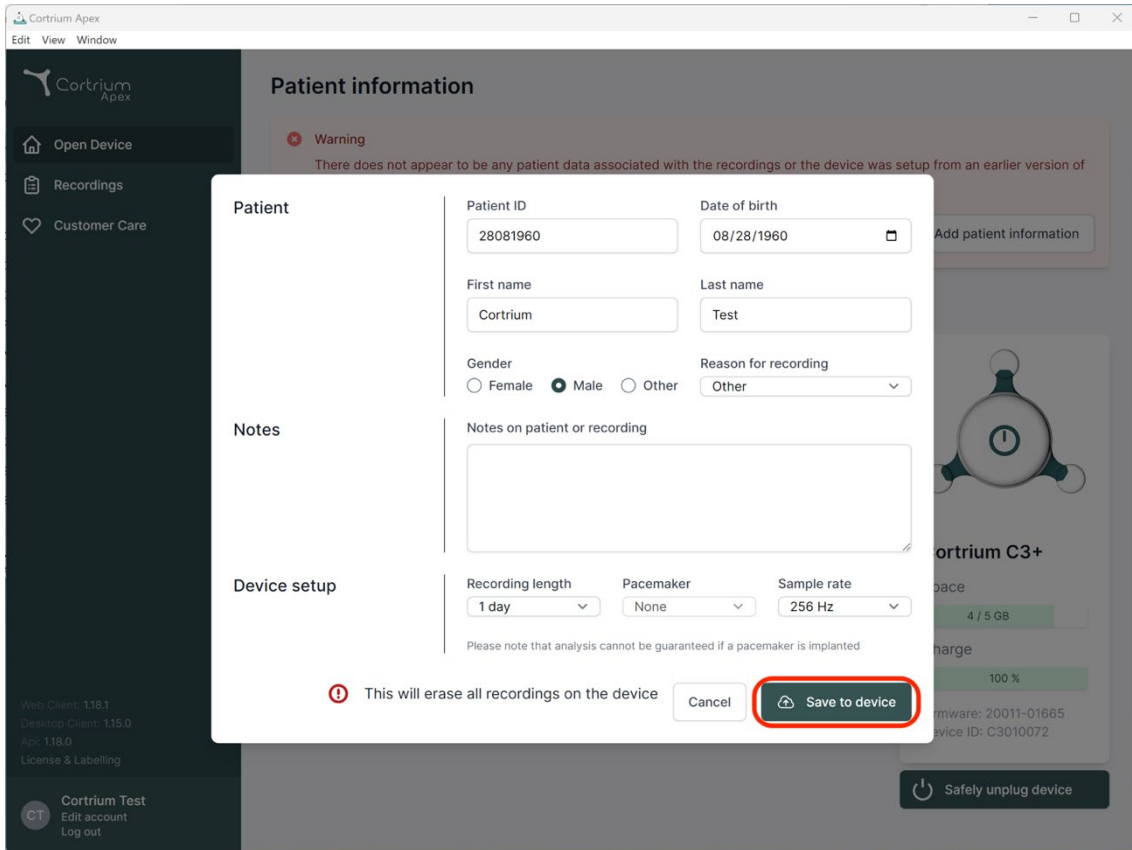
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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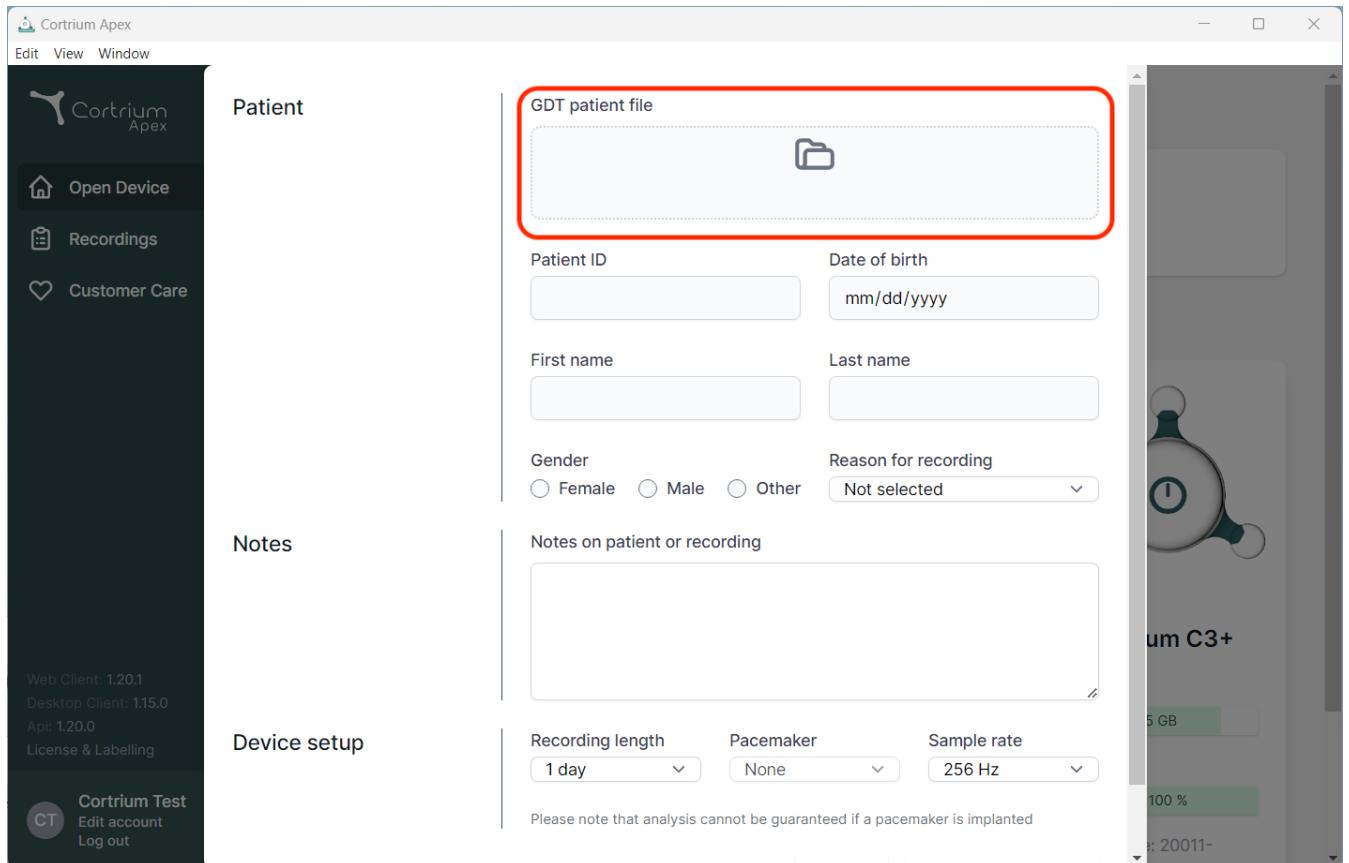
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The 'GDT patient file' field is highlighted with a red box. Below it are fields for Patient ID, Date of birth, First name, Last name, Gender, and Reason for recording. The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdowns for Recording length, Pacemaker, and Sample rate. A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'.

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

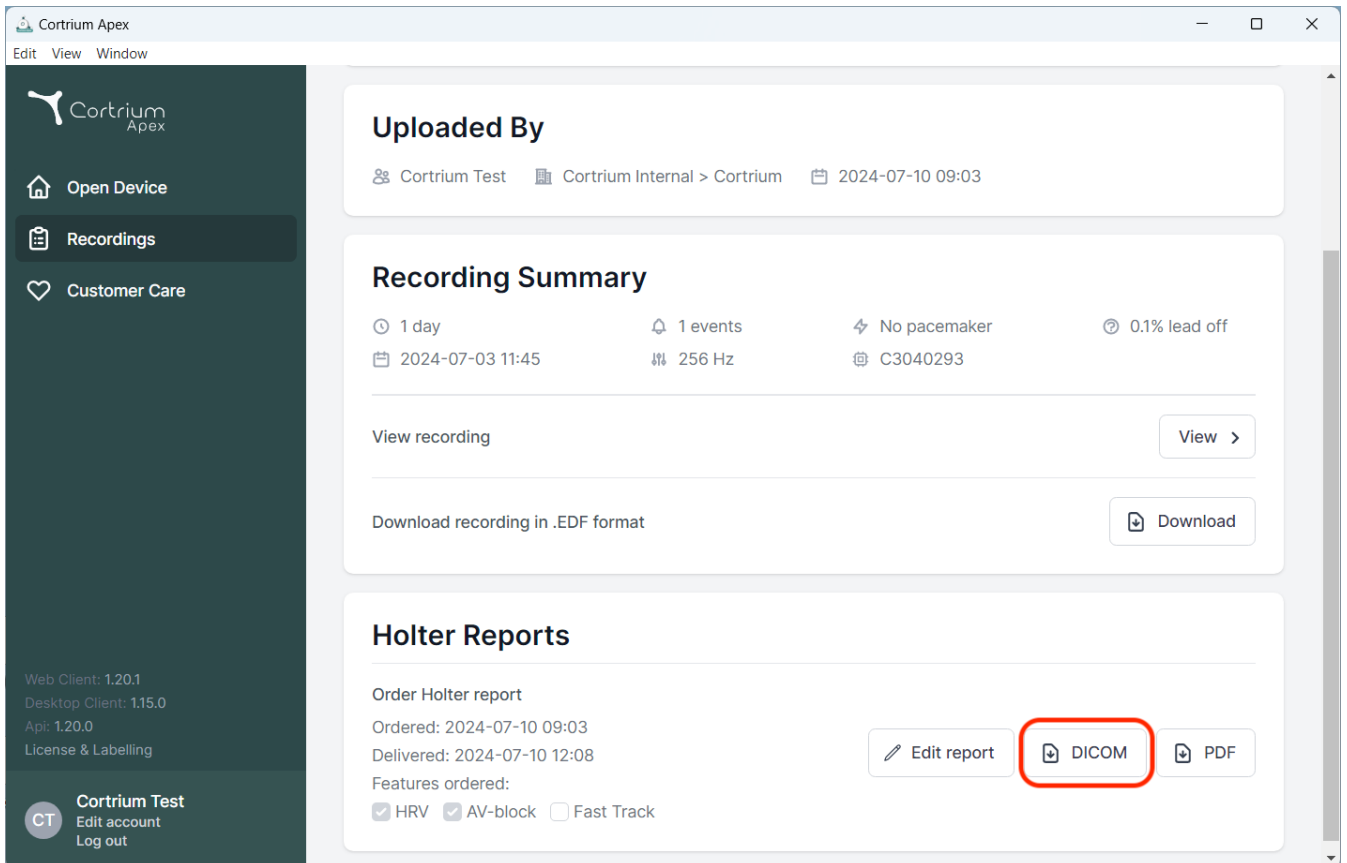
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area has a header 'Cortrium Apex' and a menu 'Edit View Window'. The 'Recordings' section displays 'Uploaded By' information: 'Cortrium Test', 'Cortrium Internal > Cortrium', and '2024-07-10 09:03'. Below this is the 'Recording Summary' section, which includes details like '1 day', '1 events', 'No pacemaker', '0.1% lead off', and '2024-07-03 11:45'. There are buttons for 'View recording' and 'Download recording in .EDF format'. The 'Holter Reports' section shows 'Order Holter report' details, including 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It also lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right of this section, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

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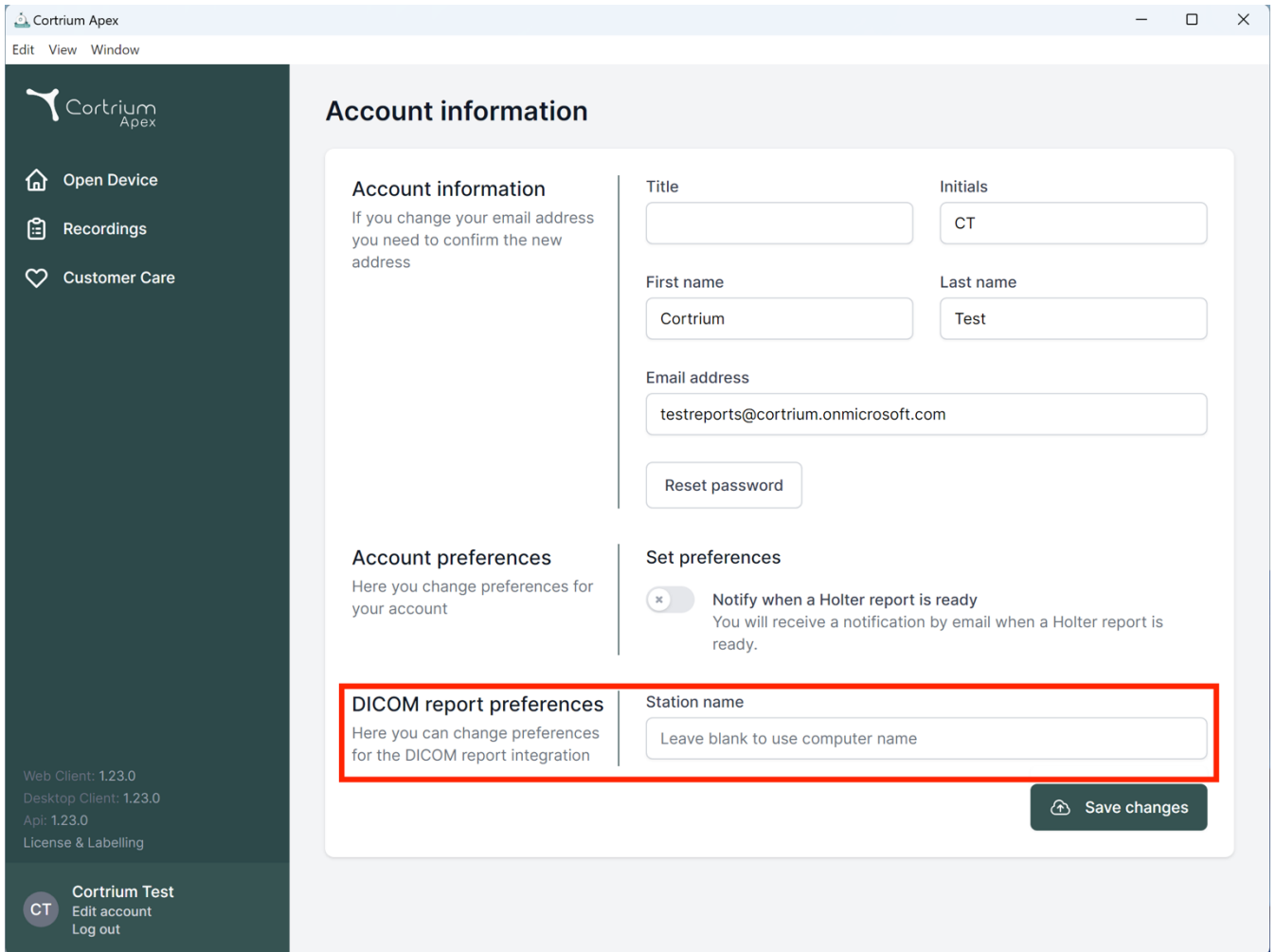The screenshot displays the 'Cortrium Apex' web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.23.0, Desktop Client: 1.23.0, Api: 1.23.0, License & Labelling) and a user profile section for 'Cortrium Test' with 'Edit account' and 'Log out' options. The main content area is titled 'Account information' and contains several sections: 'Account information' with a note about email changes, fields for Title, First name (Cortrium), Last name (Test), and Email address (testreports@cortrium.onmicrosoft.com), and a 'Reset password' button; 'Account preferences' with a note about changing preferences; 'Set preferences' with a toggle for 'Notify when a Holter report is ready'; and 'DICOM report preferences' with a note about changing preferences for DICOM integration. The 'DICOM report preferences' section is highlighted with a red border and includes a 'Station name' field with the placeholder text 'Leave blank to use computer name'. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

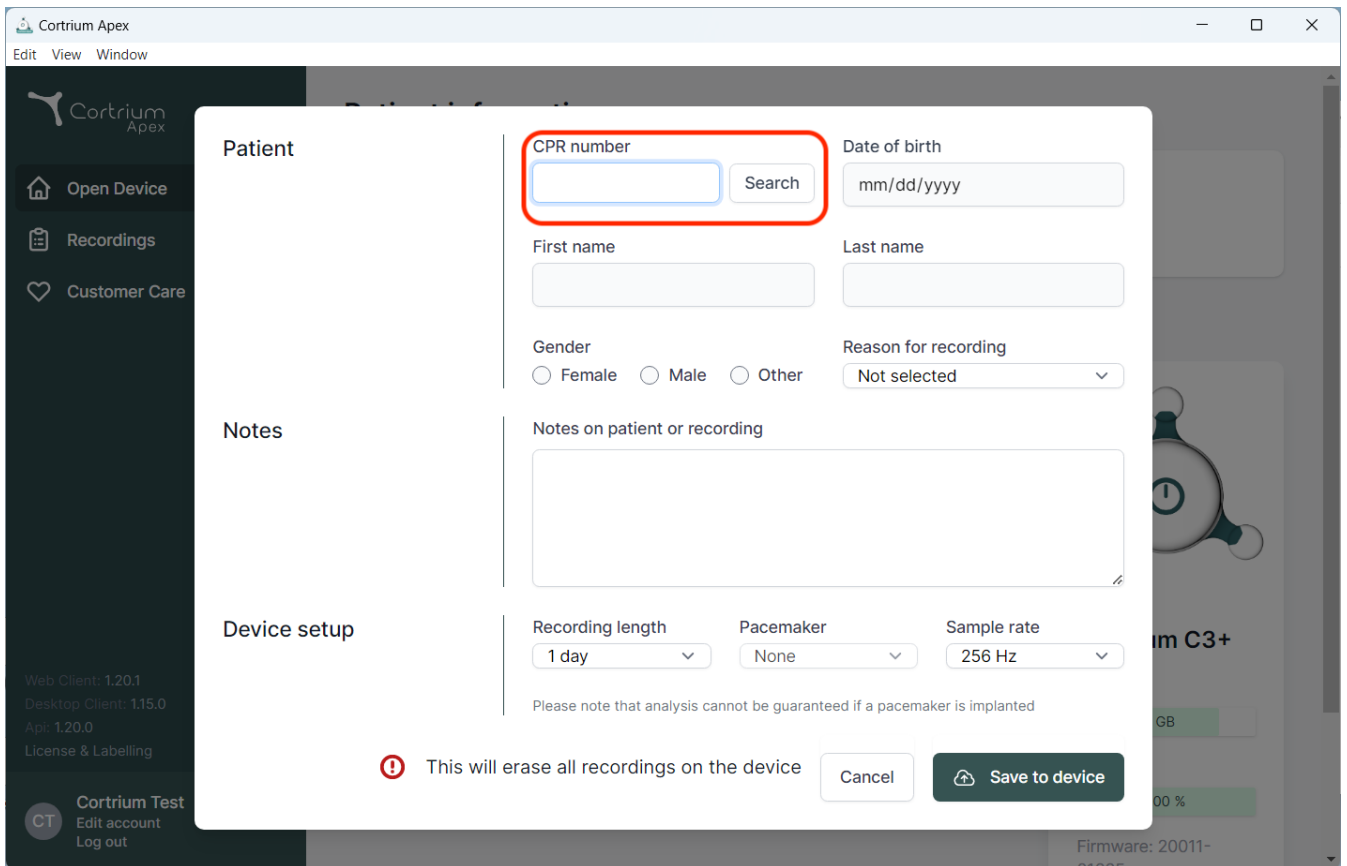
Here you can change preferences for the DICOM report integration

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The screenshot shows the Cortrium Apex web application interface. A modal dialog box titled "Patient" is open, allowing for patient information entry. The dialog is divided into sections: "Patient", "Notes", and "Device setup".

Patient Section:

- CPR number:** A text input field with a blue border, highlighted by a red rectangle, and a "Search" button next to it.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

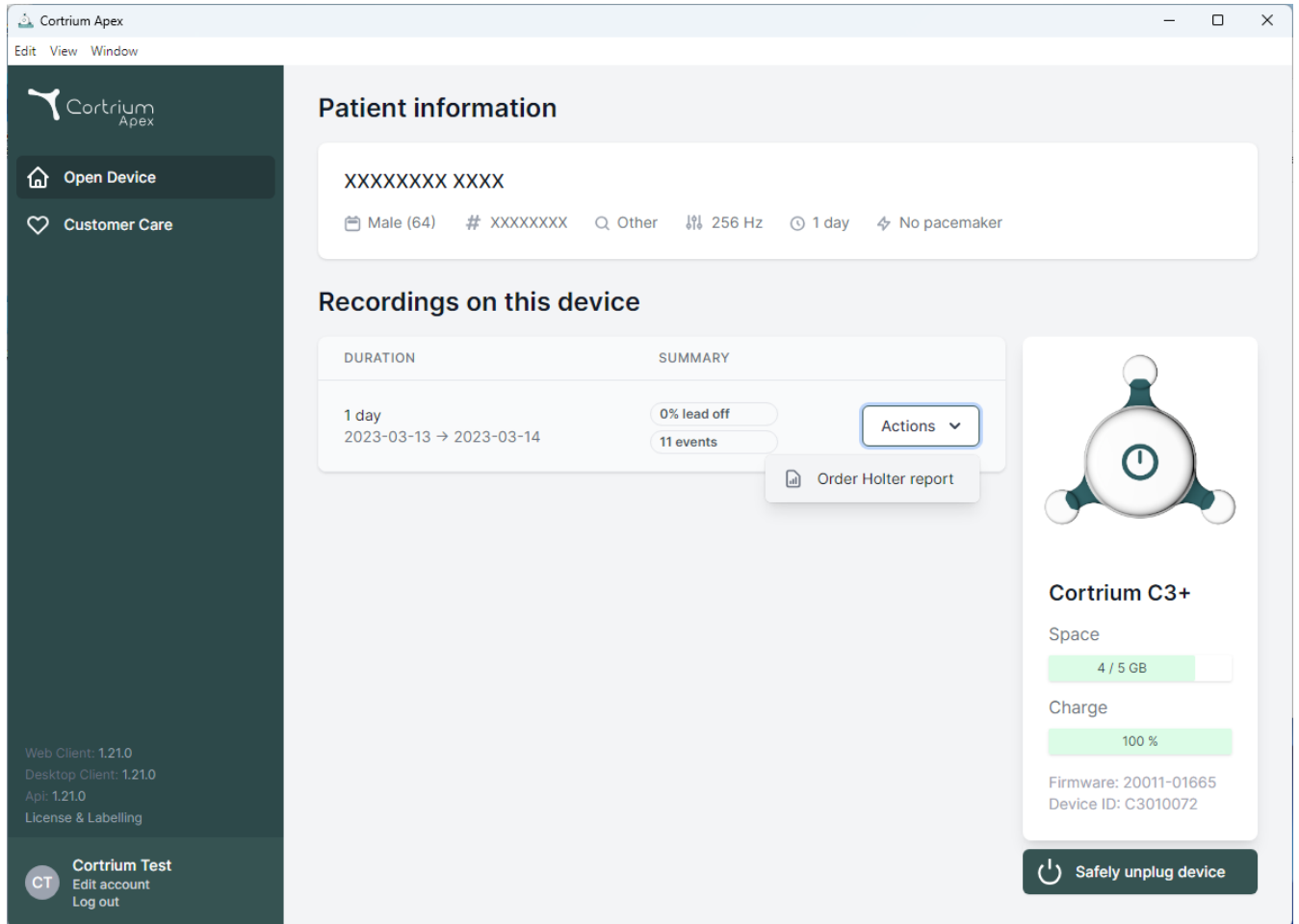
Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the dialog, there is a red warning icon and the text "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar includes version information (Web Client: 1.20.1, Desktop Client: 1.15.0, App: 1.20.0) and a user profile section for "Cortrium Test" with options to "Edit account" and "Log out".

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' options.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section displays a table with two columns: 'DURATION' and 'SUMMARY'. The table shows a recording for '1 day' from '2023-03-13' to '2023-03-14', with '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible, with 'Order Holter report' as an option. On the right side, a device status card for 'Cortrium C3+' shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of this card.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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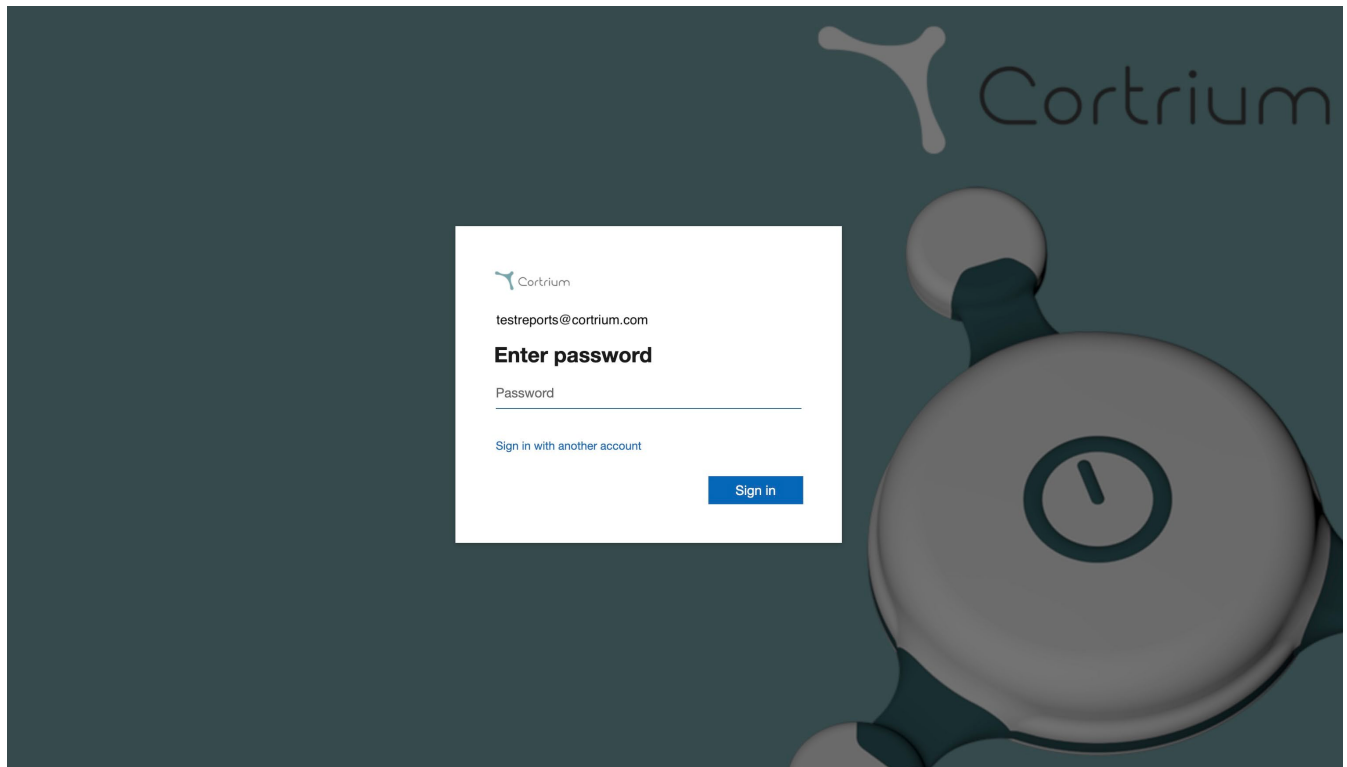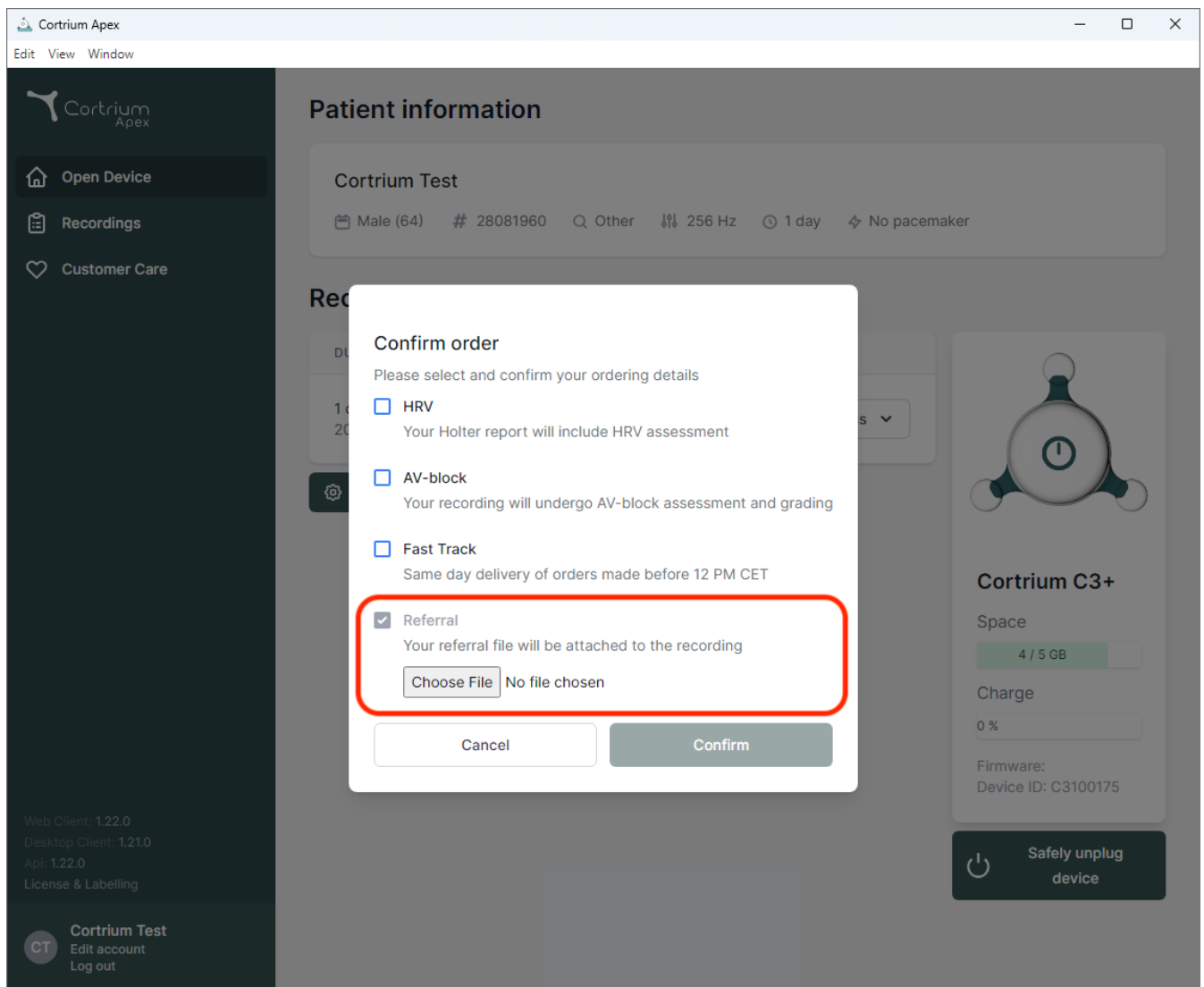
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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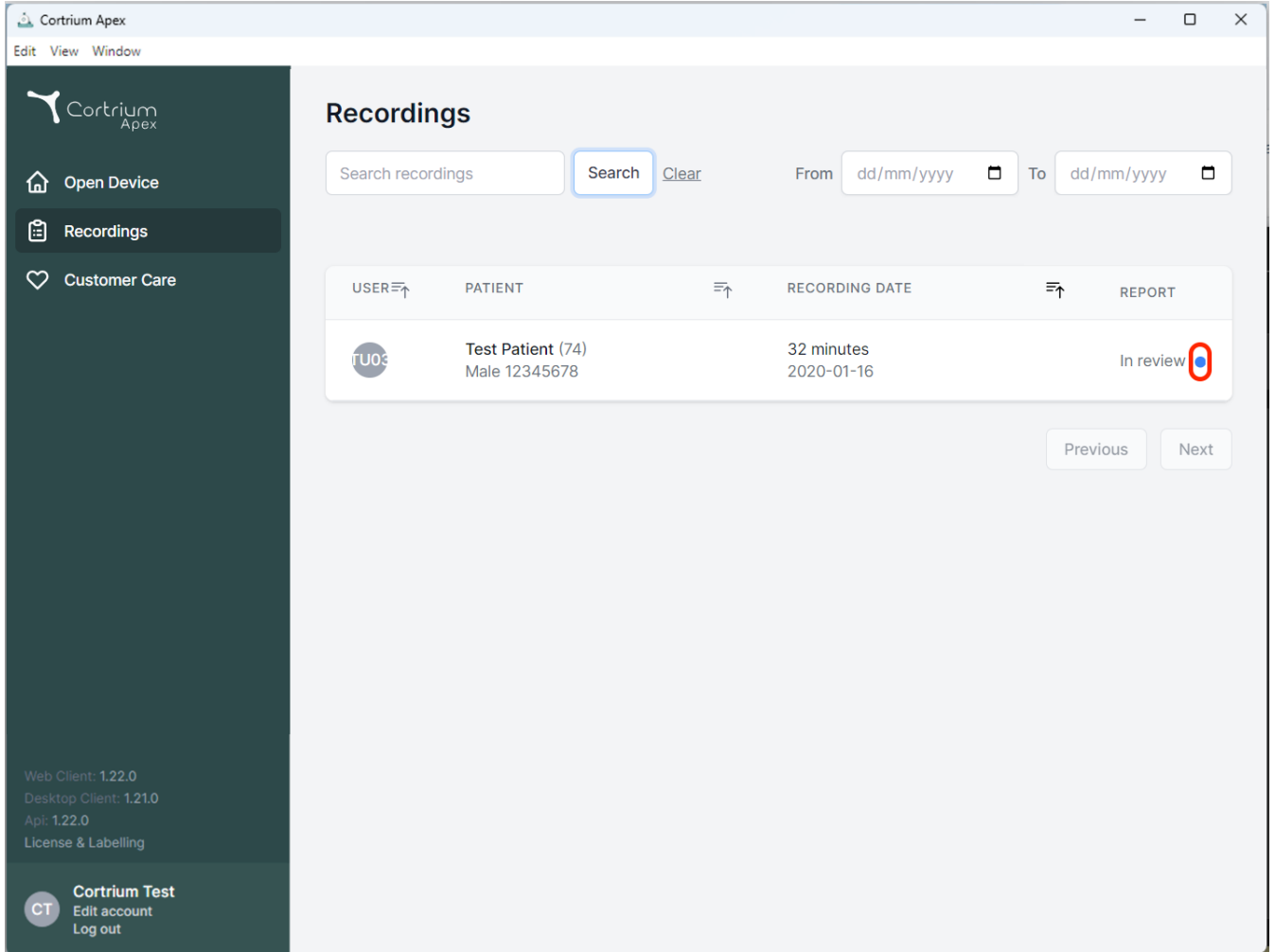
Search recordings Search Clear


From dd/mm/yyyy To dd/mm/yyyy

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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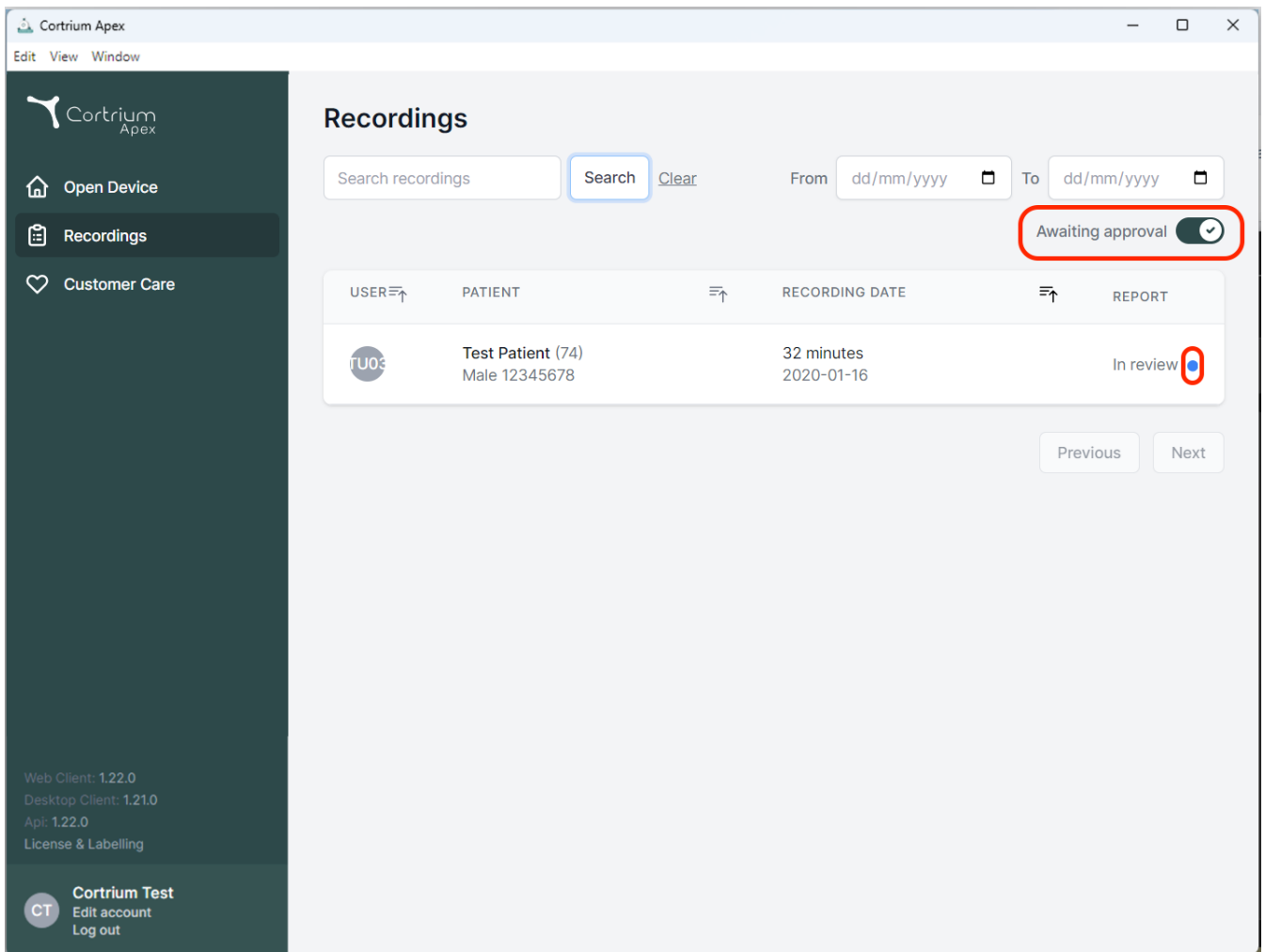Recordings

Search recordings **Search** [Clear](#) From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
U03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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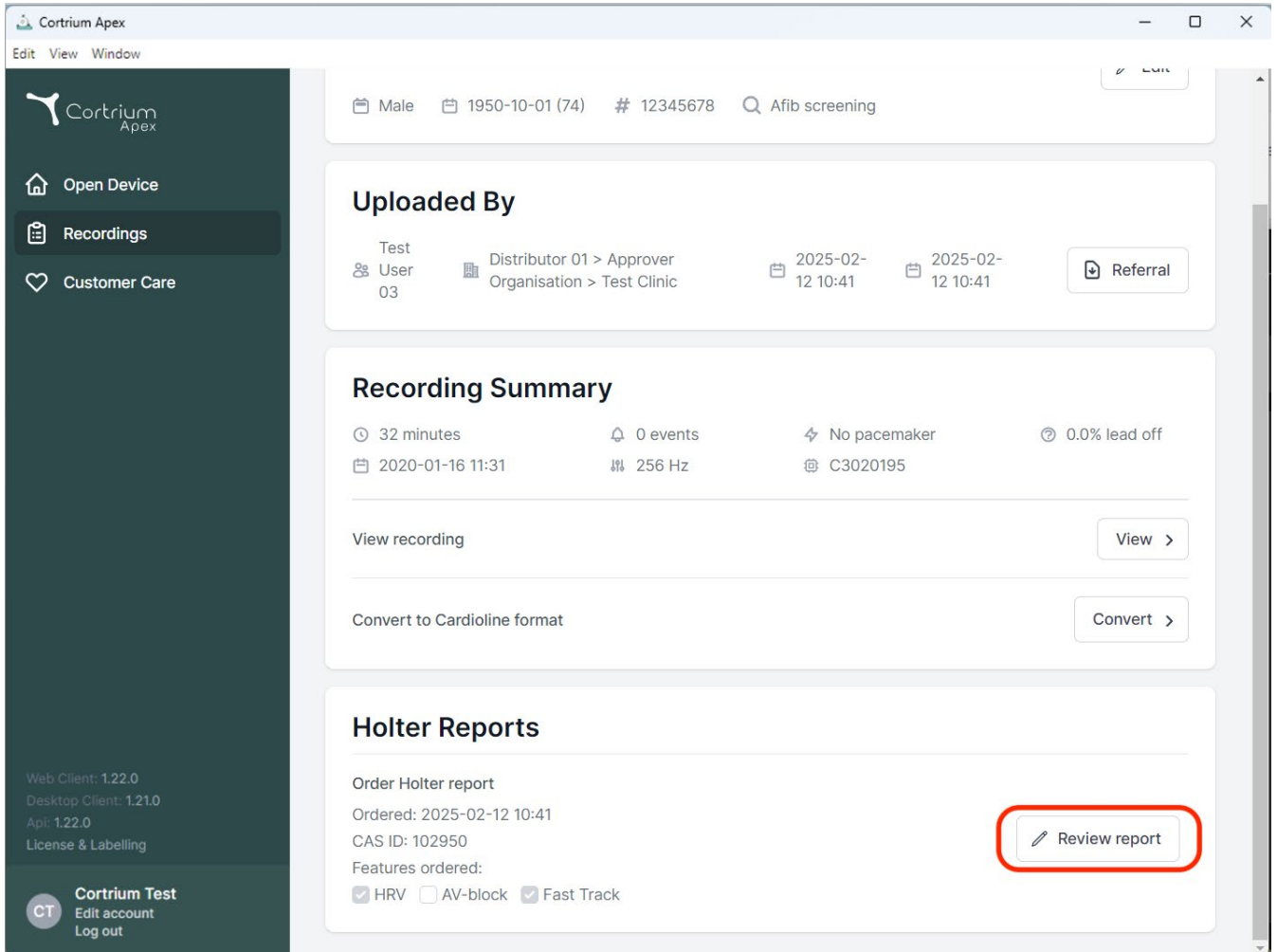The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes the Cortrium Apex logo and a menu with 'Edit', 'View', and 'Window'. The left sidebar contains navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the 'Uploaded By' section, showing 'Test User 03' as the distributor, 'Distributor 01 > Approver Organisation > Test Clinic' as the approver, and upload dates of 2025-02-12 10:41. A 'Referral' button is present. The 'Recording Summary' section displays: 32 minutes duration, 0 events, No pacemaker, 0.0% lead off, recording date 2020-01-16 11:31, 256 Hz frequency, and device ID C3020195. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report' with an order date of 2025-02-12 10:41, CAS ID 102950, and features ordered: HRV (checked), AV-block (unchecked), and Fast Track (checked). A 'Review report' button is highlighted with a red rectangle.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

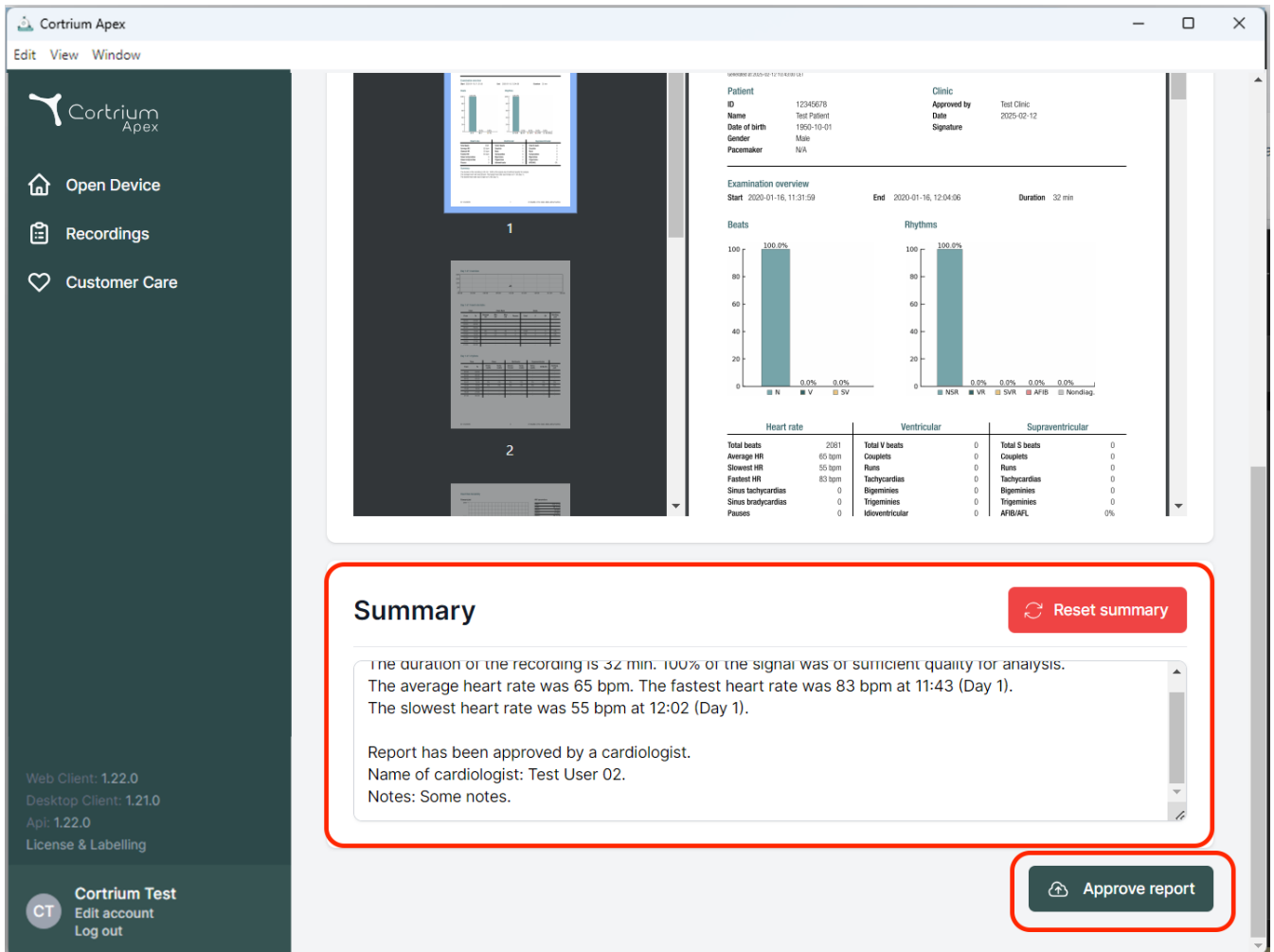
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

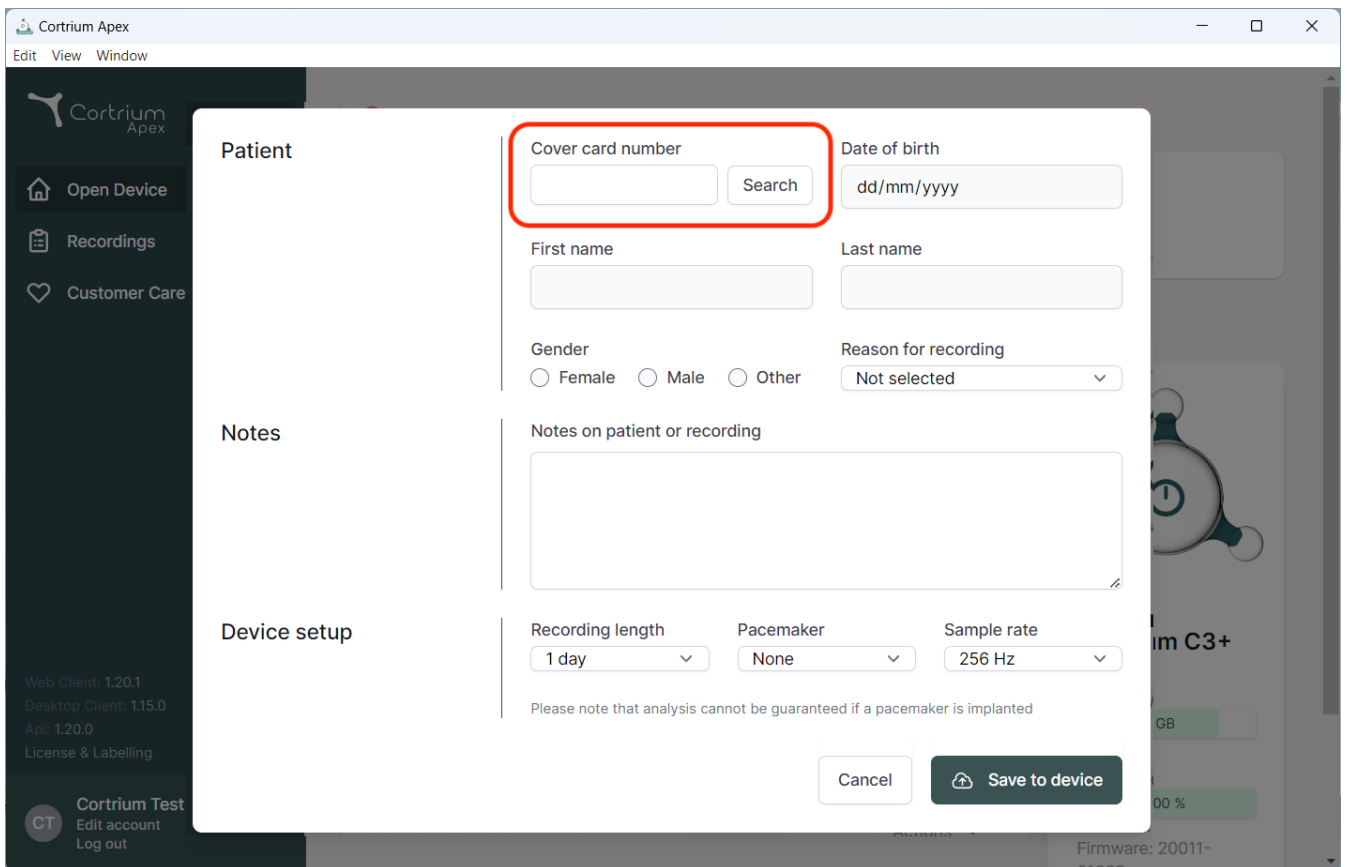
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark
E-post: info@cortrium.com • Nettsted: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
46 / 46