Cortrium Apex 1.26

Cortrium Apex 1.26

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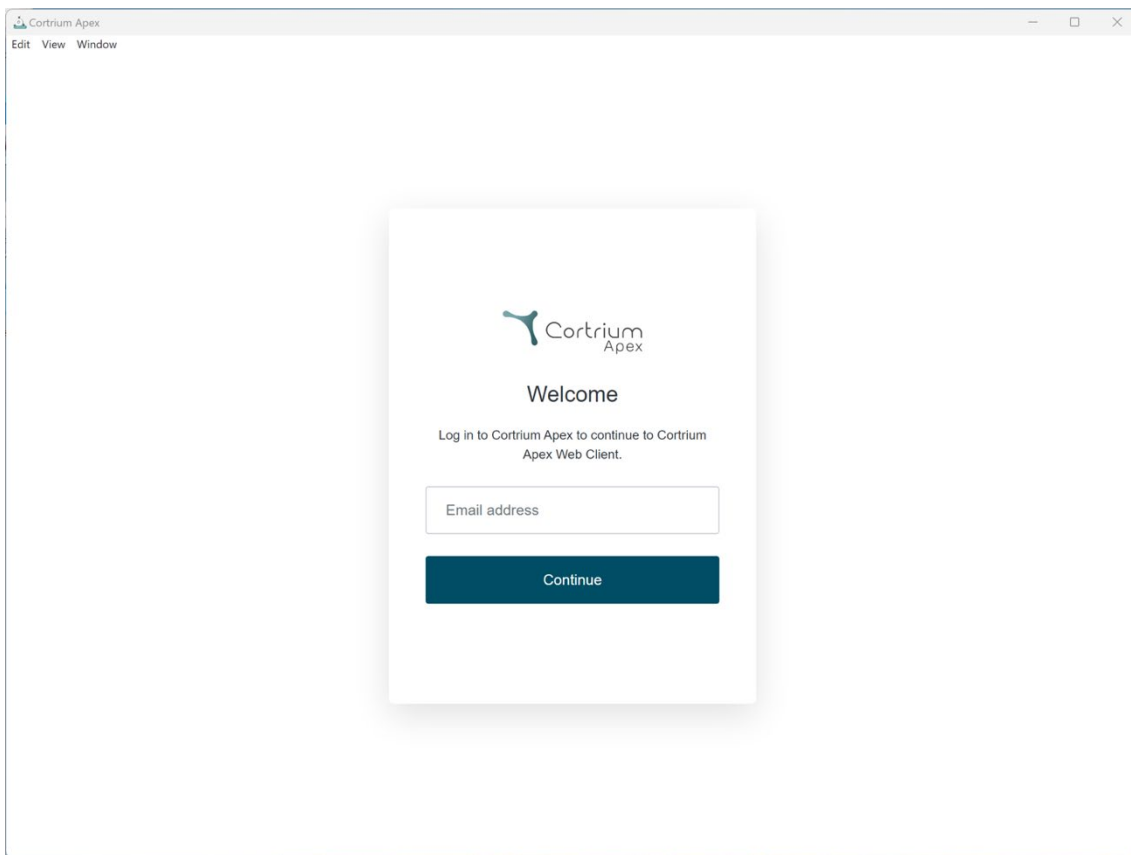
 Cortrium
Apex

Welcome

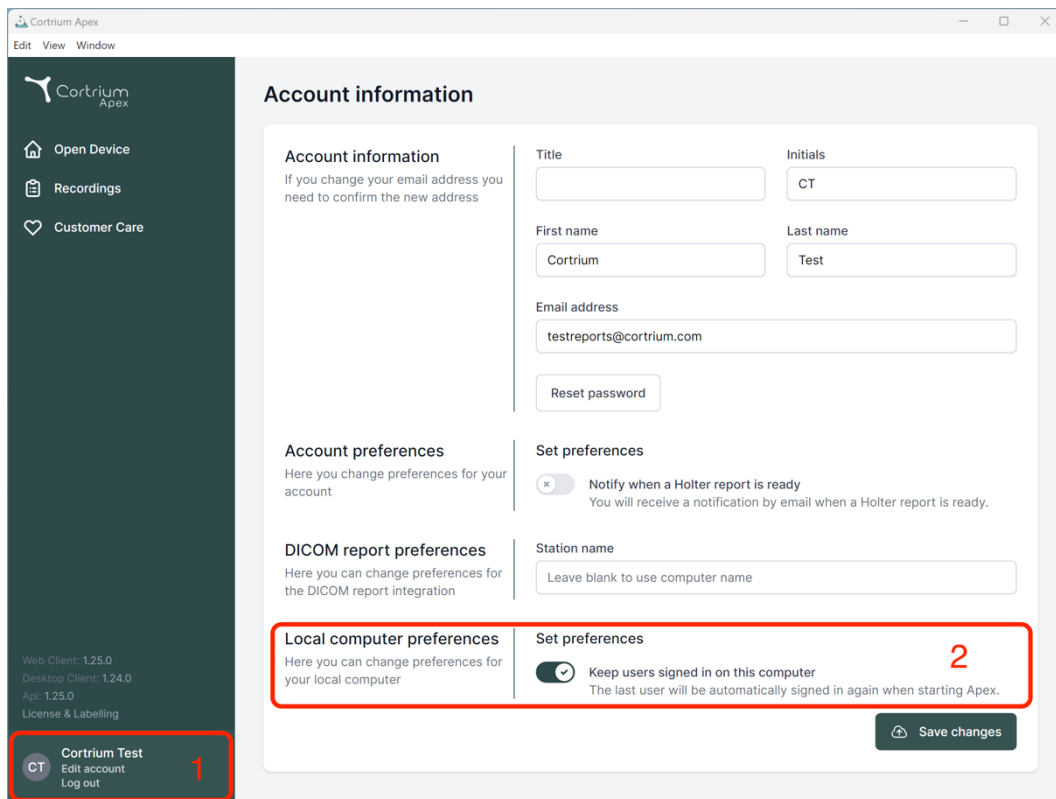
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these, it lists version information: 'Web Client: 1.25.0', 'Desktop Client: 1.24.0', 'Api: 1.25.0', and 'License & Labelling'. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address), 'Account preferences' (with a toggle for 'Notify when a Holter report is ready'), 'DICOM report preferences' (with a field for 'Station name'), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer'). The 'Local computer preferences' section is highlighted with a red box and a red '2'. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

Account preferences

Here you change preferences for your account

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DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

Local computer preferences

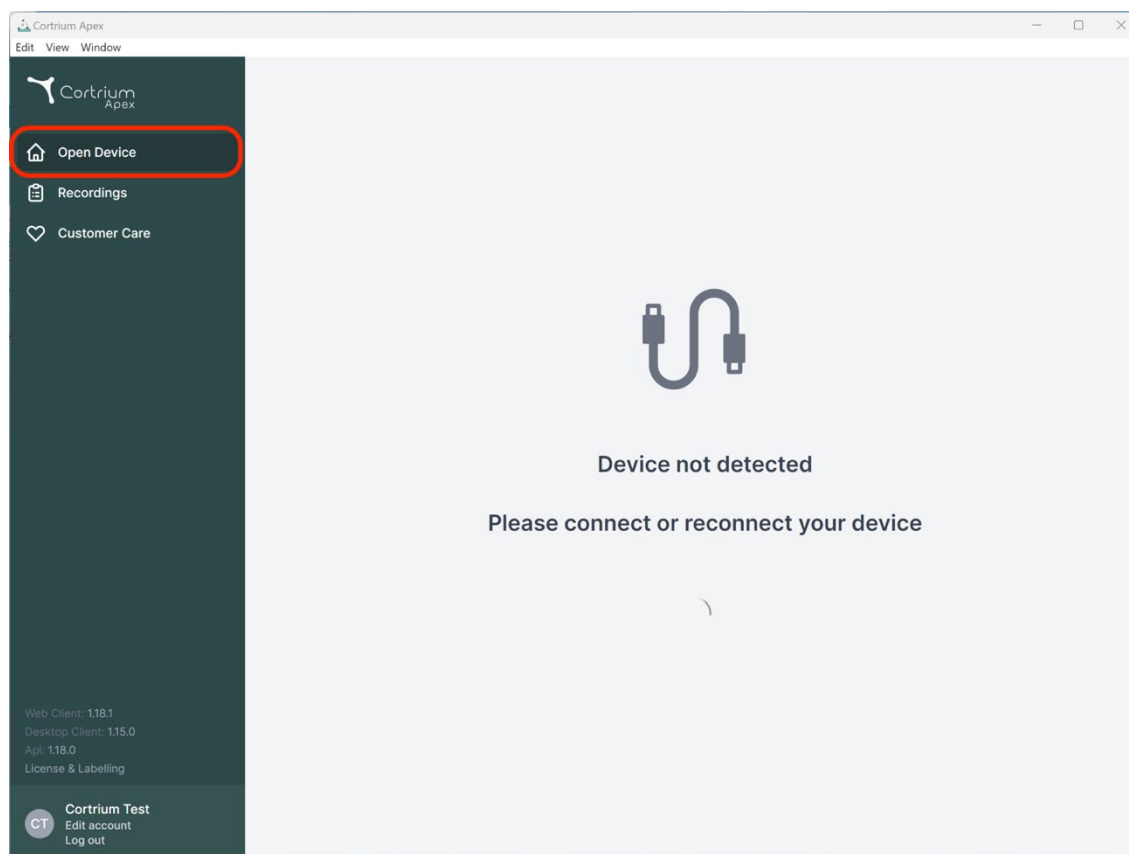
Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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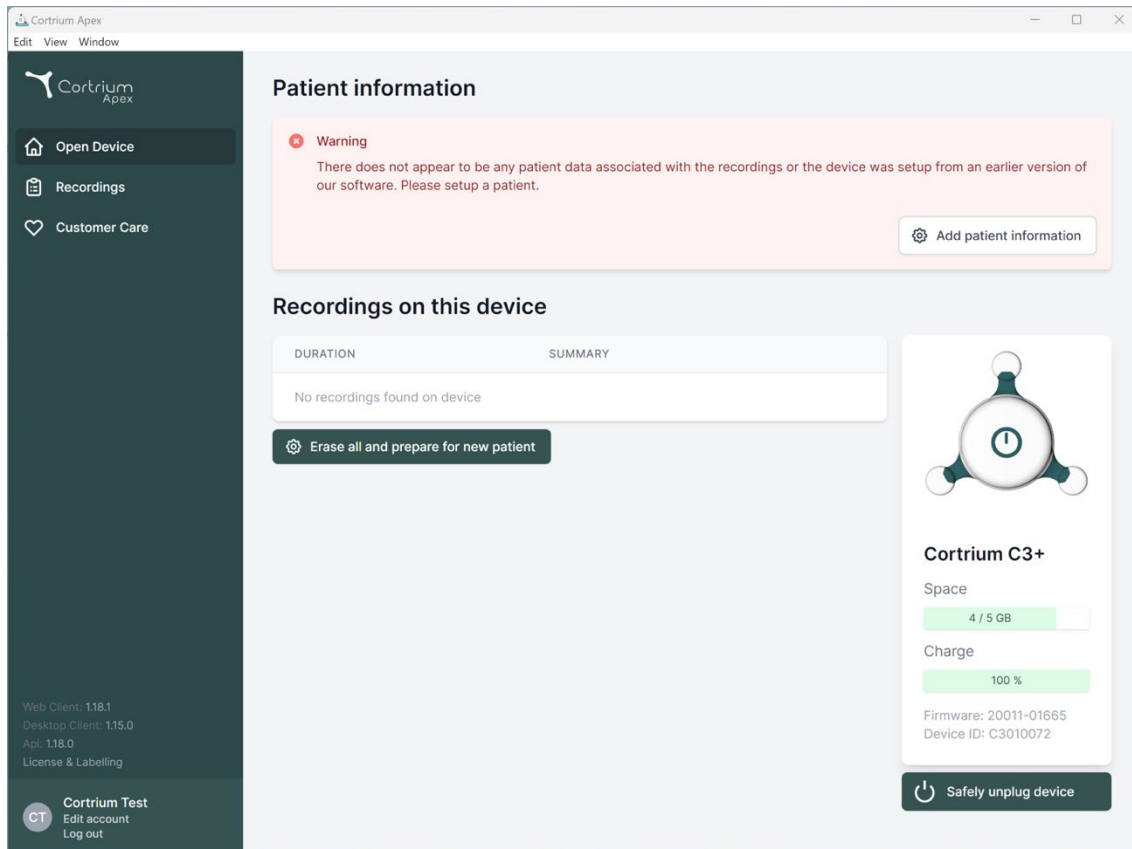
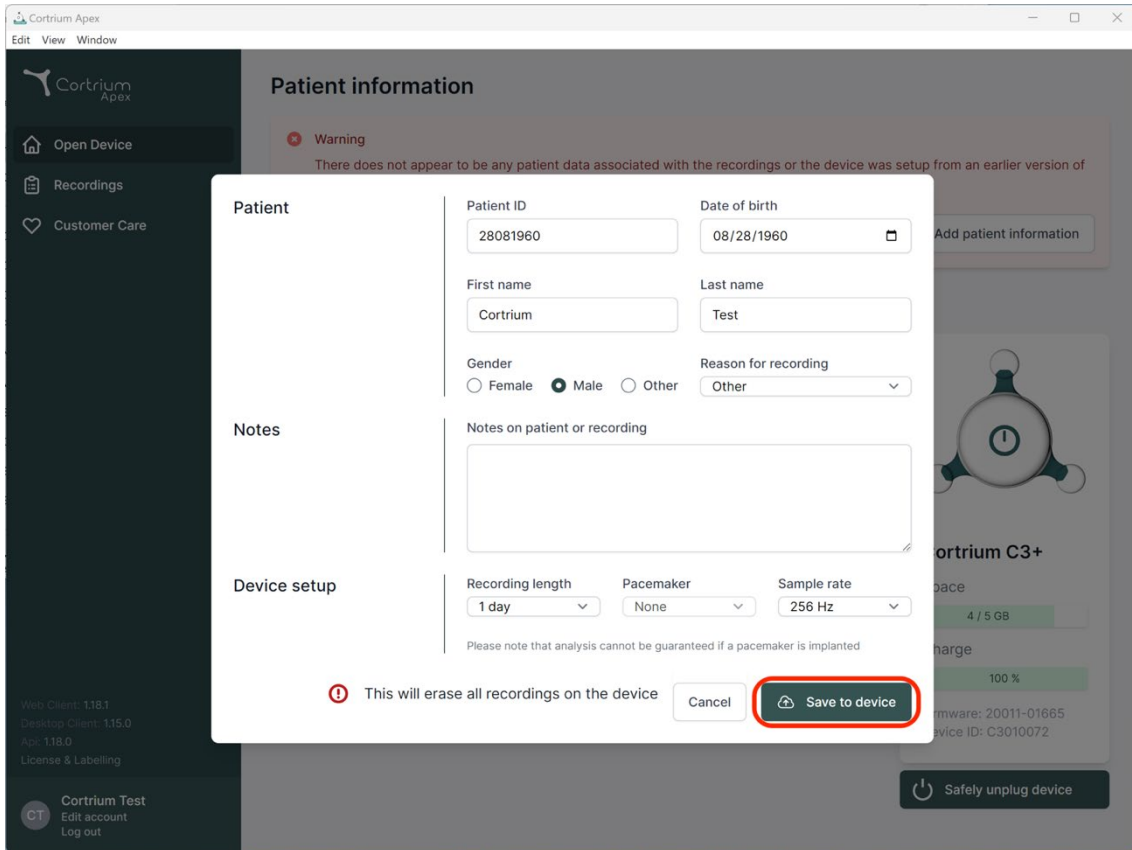
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
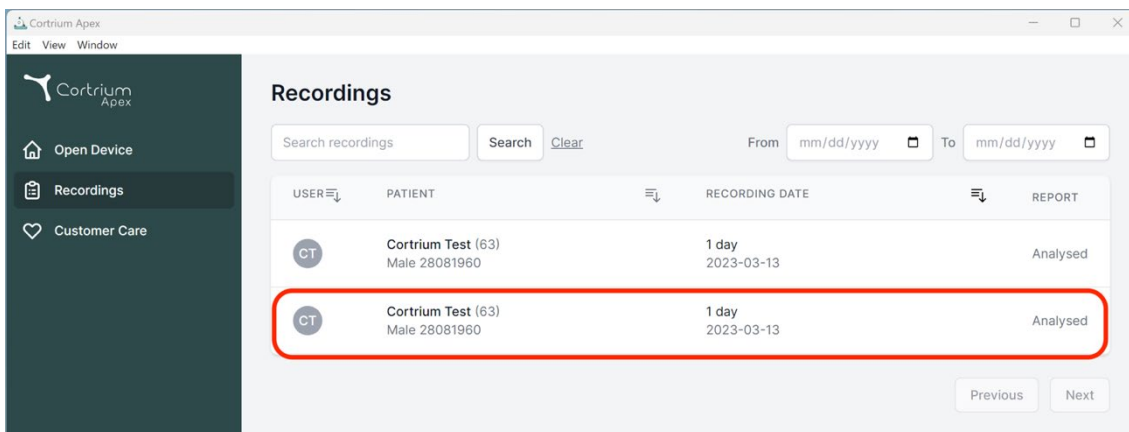
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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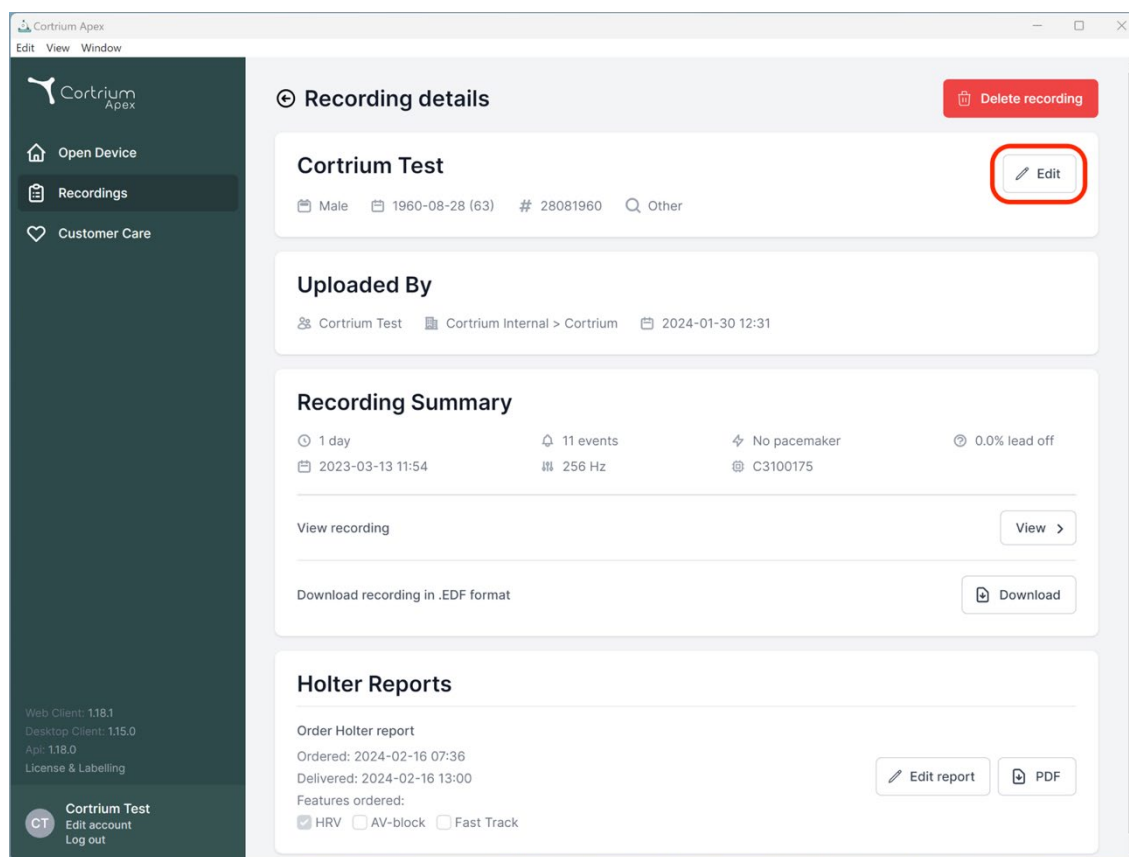2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
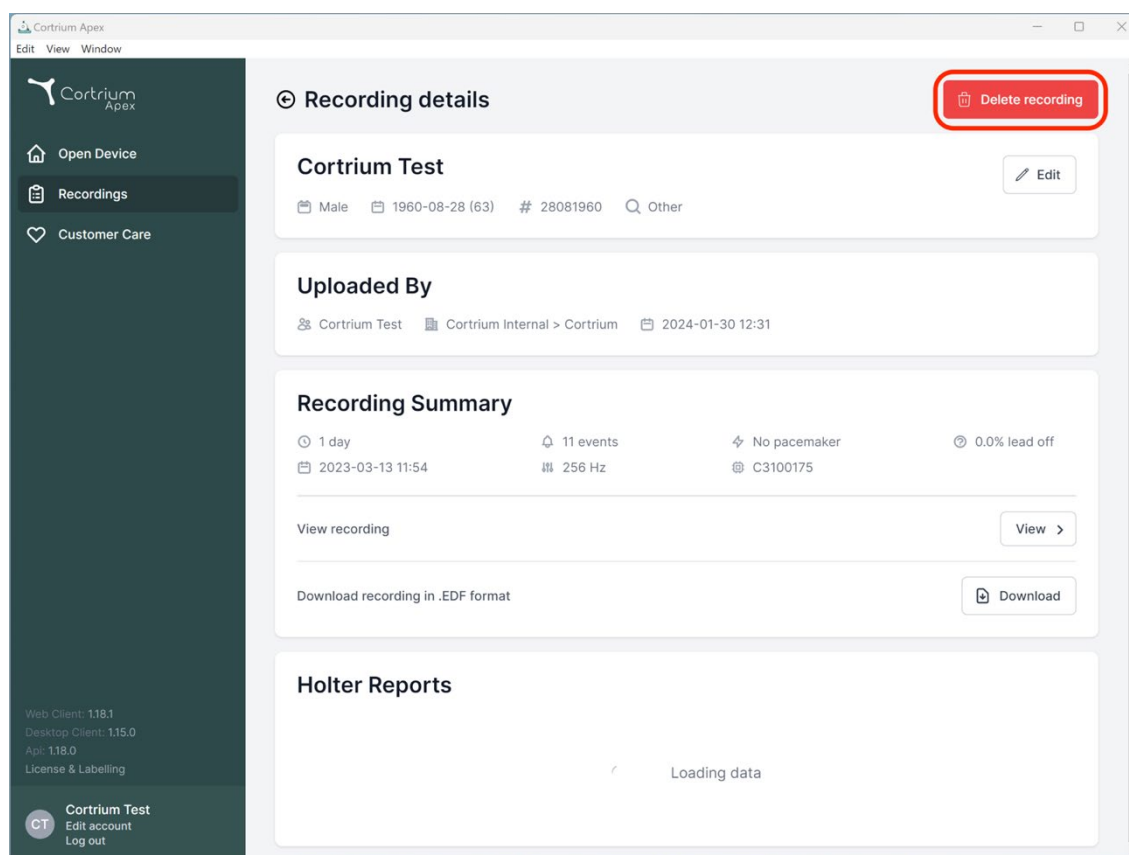
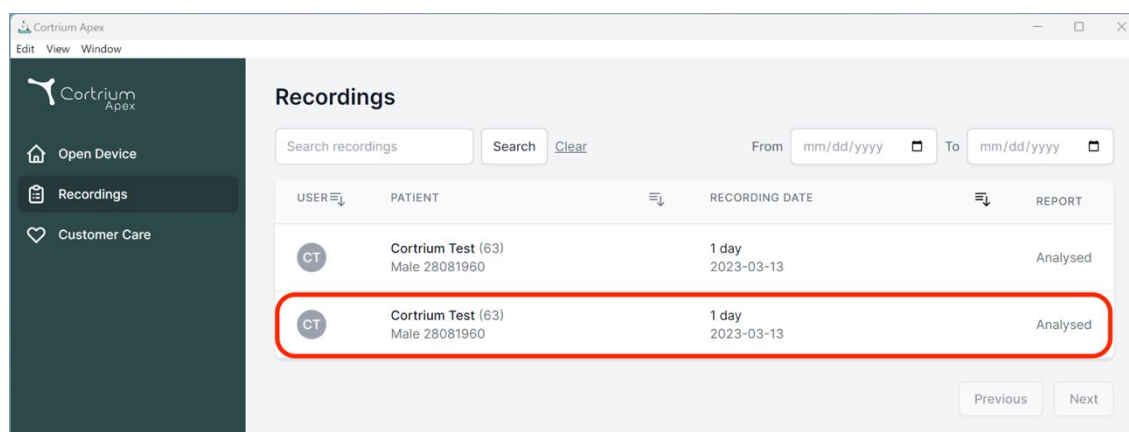
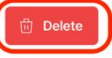
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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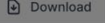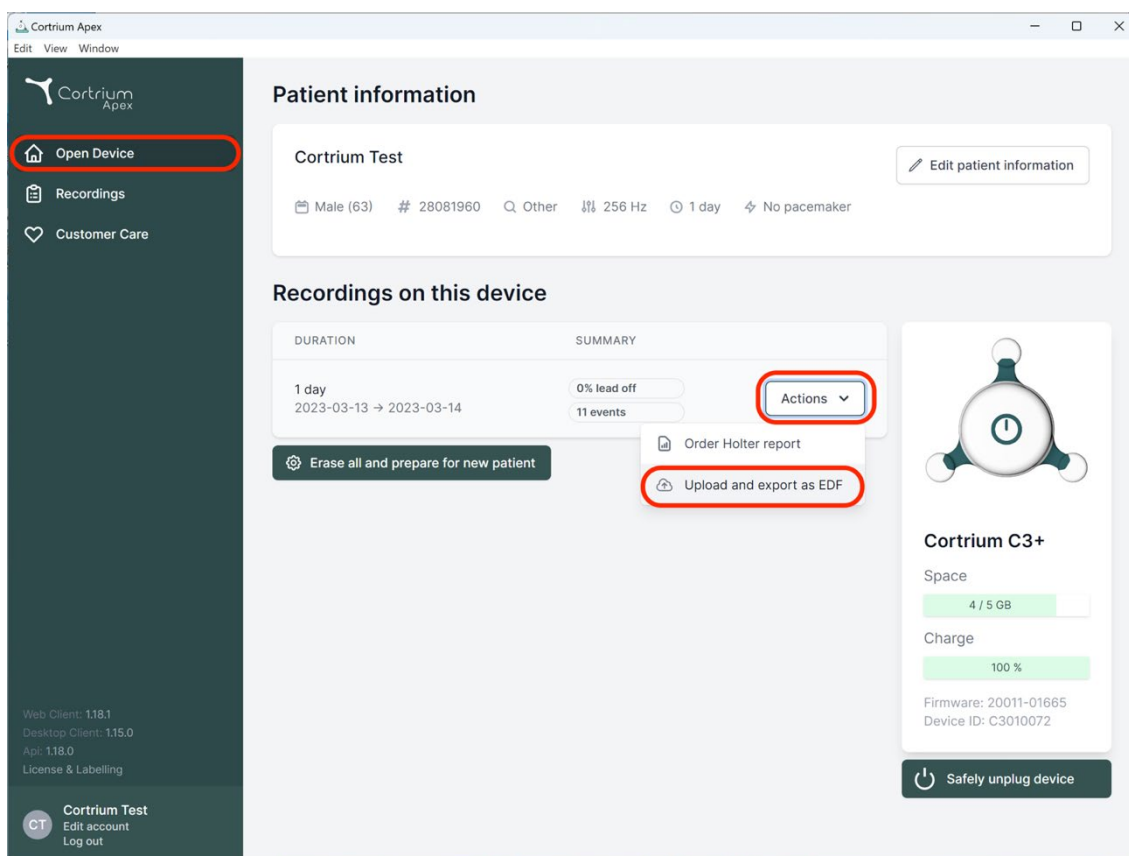
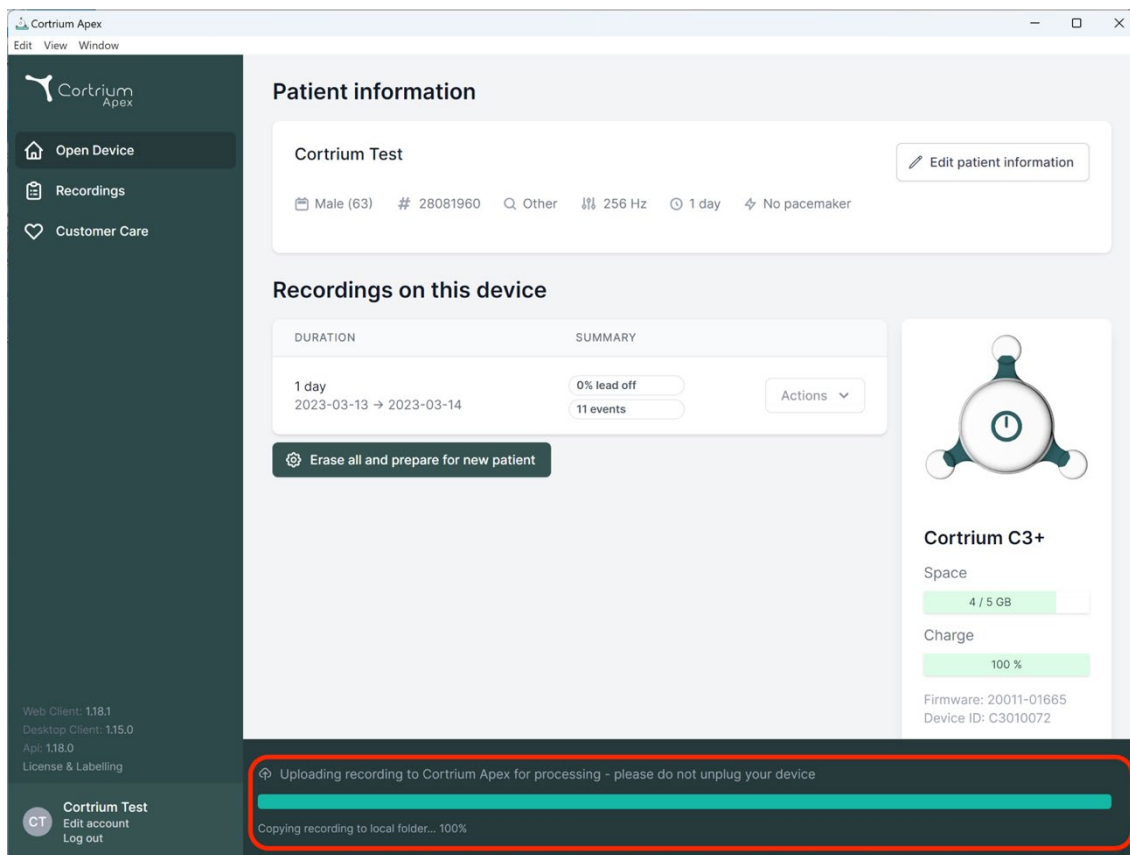
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

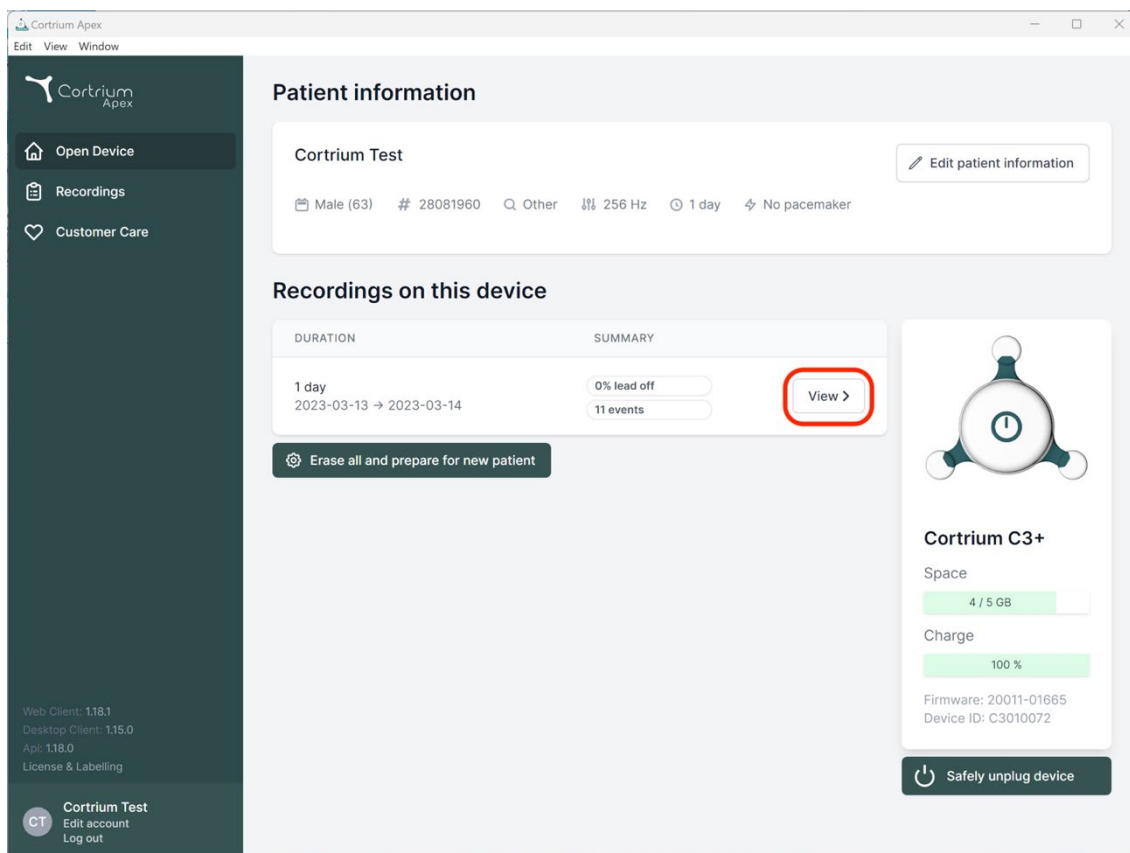
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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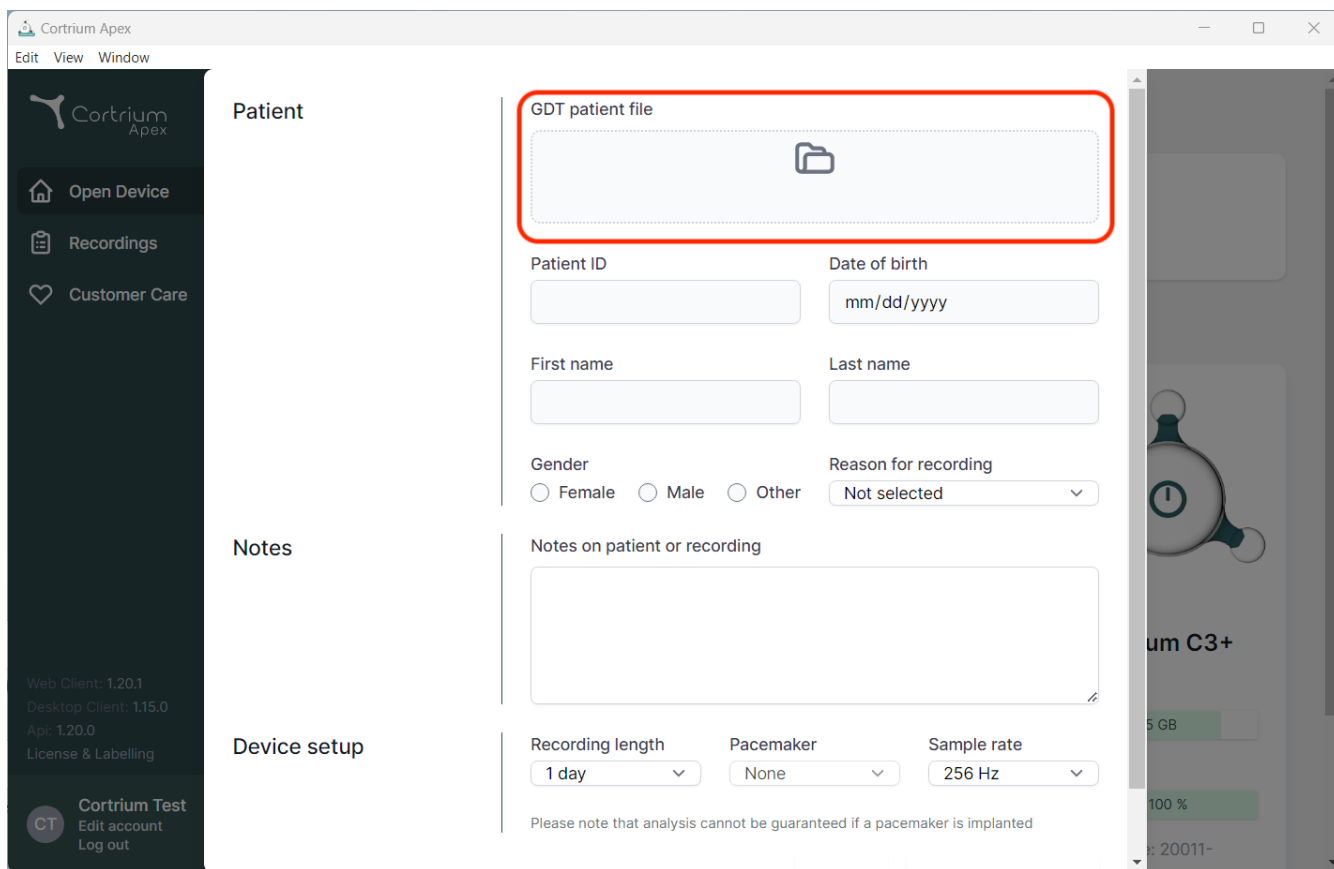
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

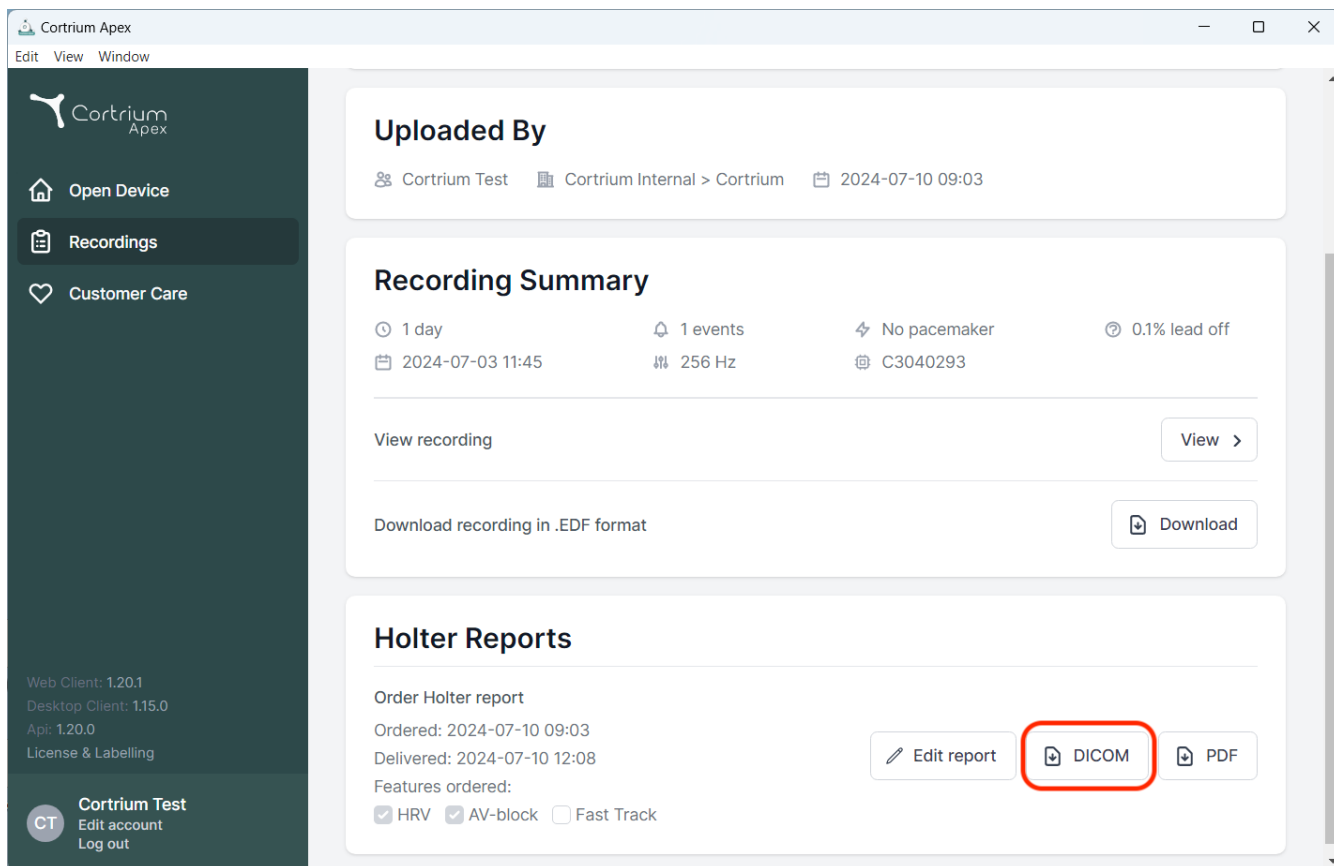
The main content area is divided into three sections:

- Patient:** Contains a "GDT patient file" field with a folder icon, highlighted by a red rectangle. Below it are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown for "Reason for recording" (currently "Not selected").
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Includes dropdowns for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). Below these is a disclaimer: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a heart icon and the text "um C3+", "5 GB", "100 %", and "20011-".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying device details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (which is highlighted with a red rectangle), and 'PDF'. Below these buttons, the 'Features ordered' section shows checkboxes for 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked).

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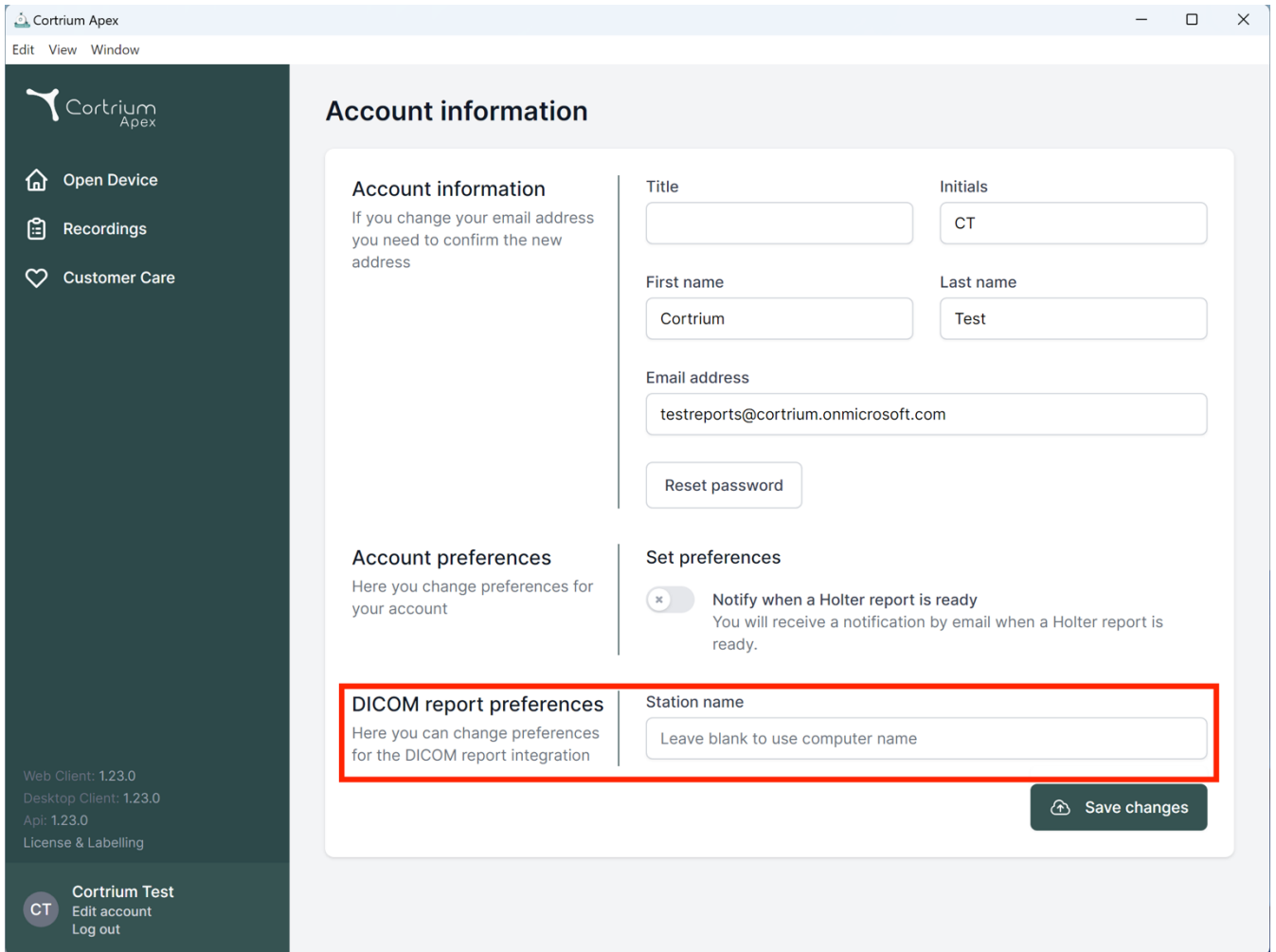Edit View Window

Account information

Account information
If you change your email address you need to confirm the new address

Title

Initials

First name

Last name

Email address

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name

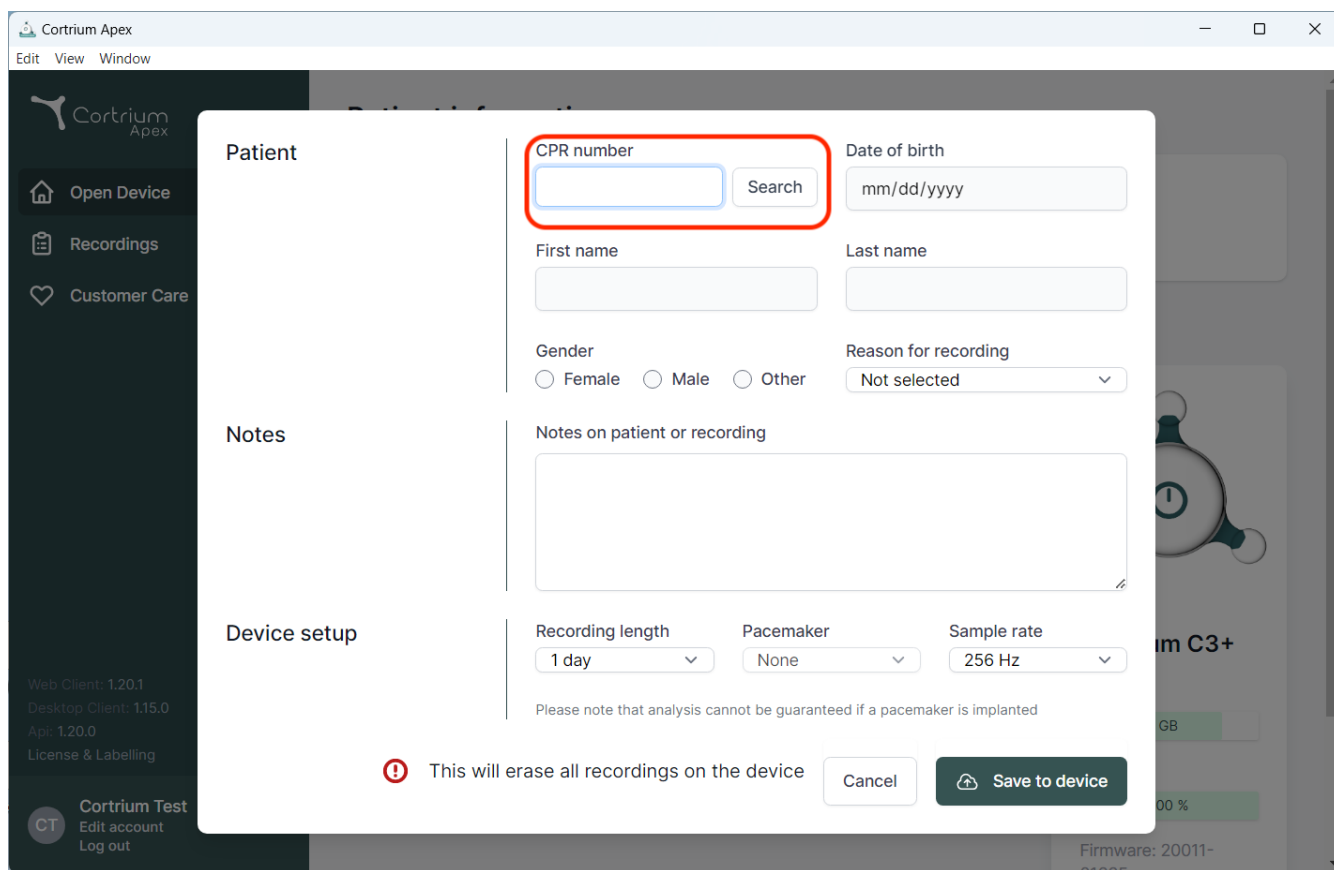
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot displays the Cortrium Apex web application interface. A modal window titled 'Patient' is open, showing a form for patient search and data entry. The 'CPR number' field is highlighted with a red box, and a 'Search' button is next to it. Other fields include 'Date of birth' (mm/dd/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), 'Reason for recording' (Not selected), 'Notes on patient or recording', 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A warning message at the bottom states: 'This will erase all recordings on the device'. The background shows the Cortrium Apex sidebar with options like 'Open Device', 'Recordings', and 'Customer Care'.

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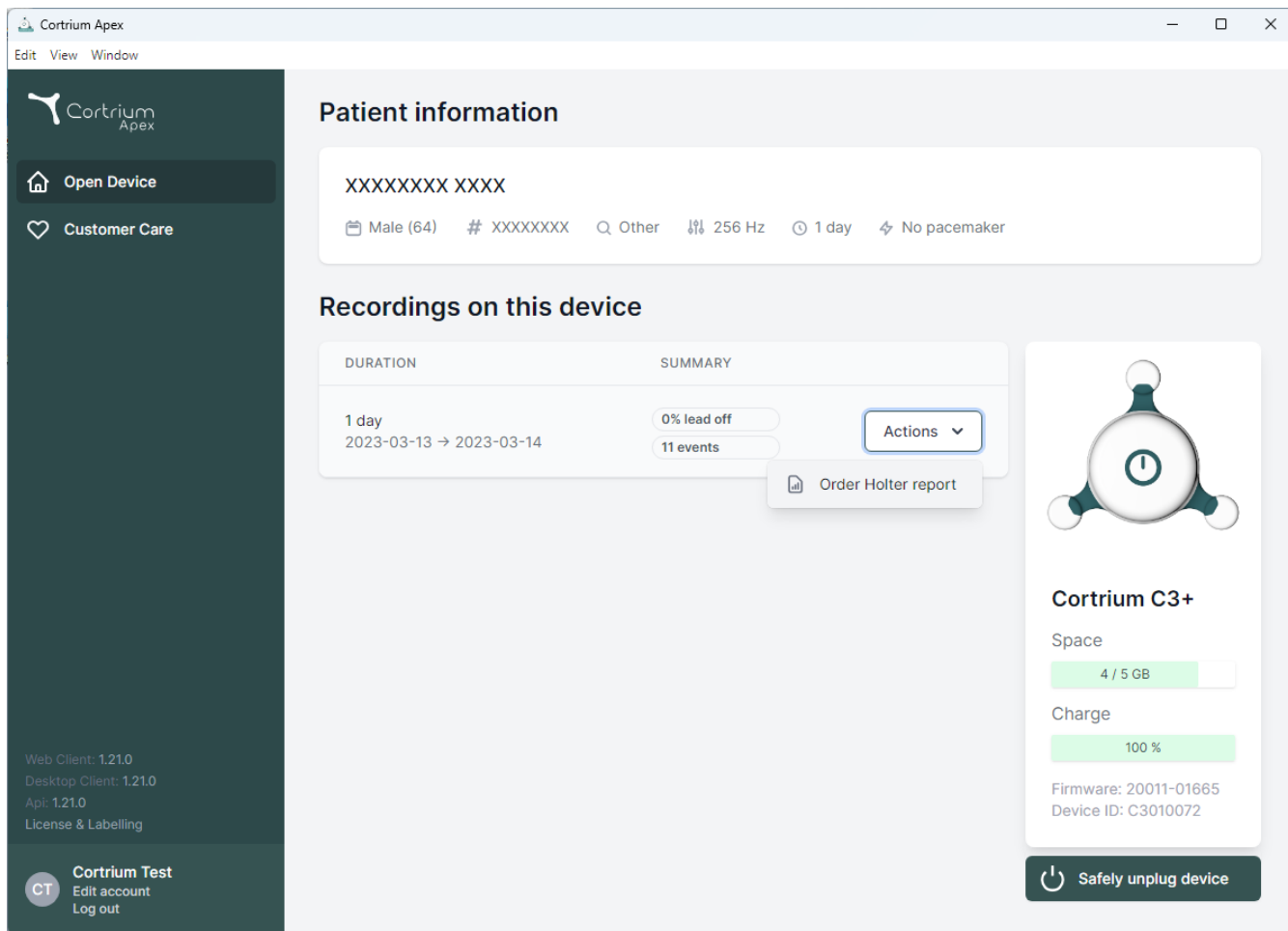
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for a '1 day' recording from '2023-03-13 → 2023-03-14'. The summary for this recording shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side of the interface, there is a section for the 'Cortrium C3+' device, showing its status: 'Space' (4 / 5 GB), 'Charge' (100%), 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of this section.

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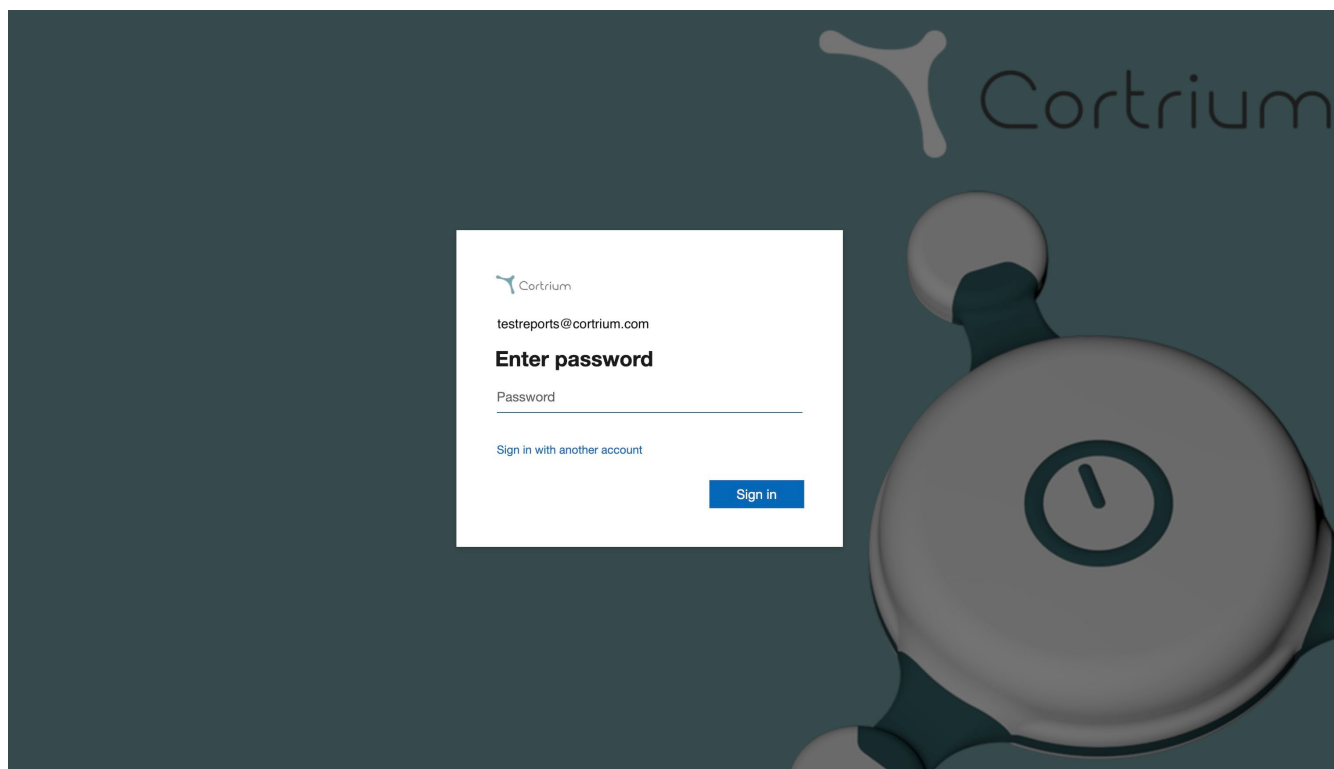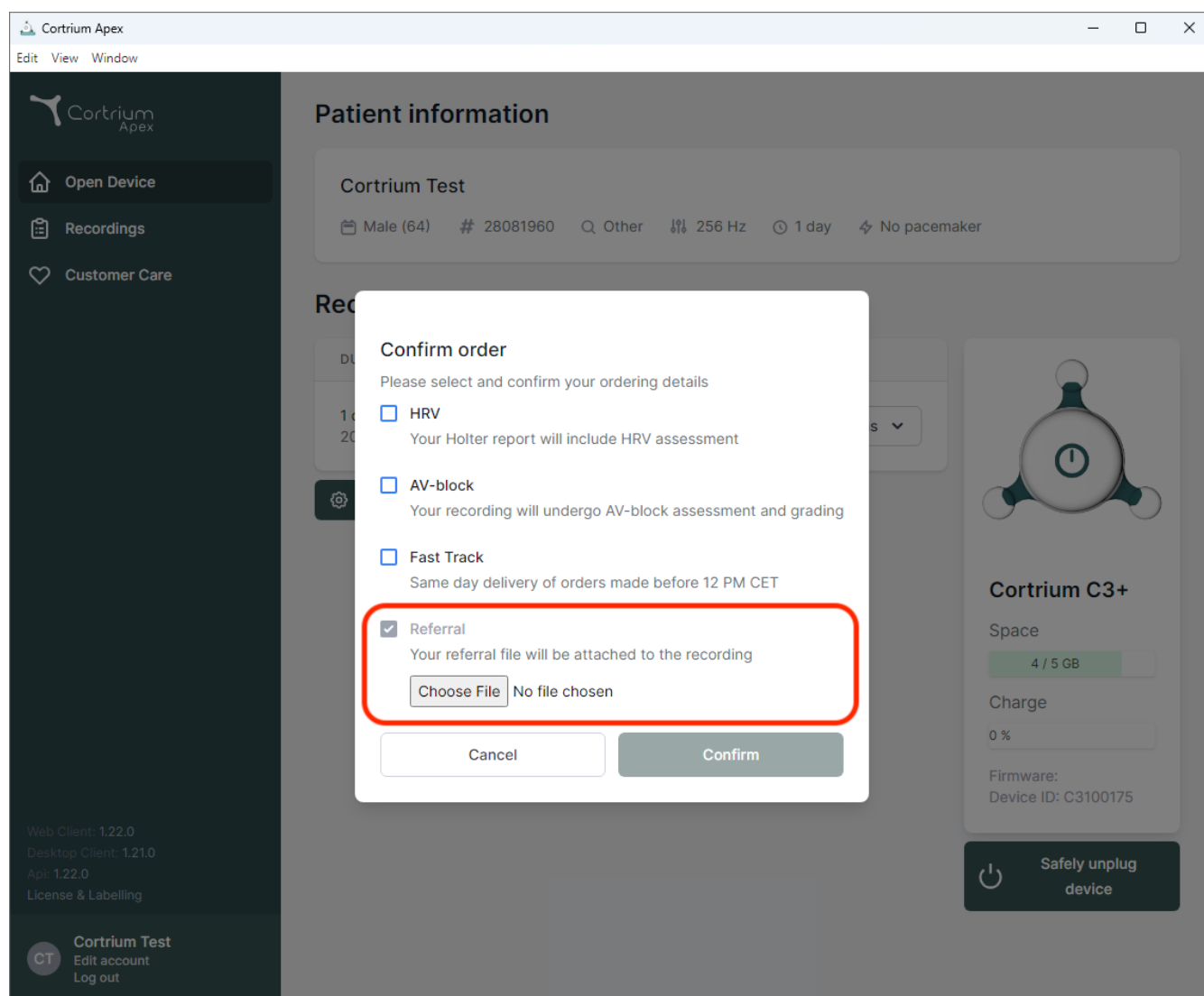
Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off

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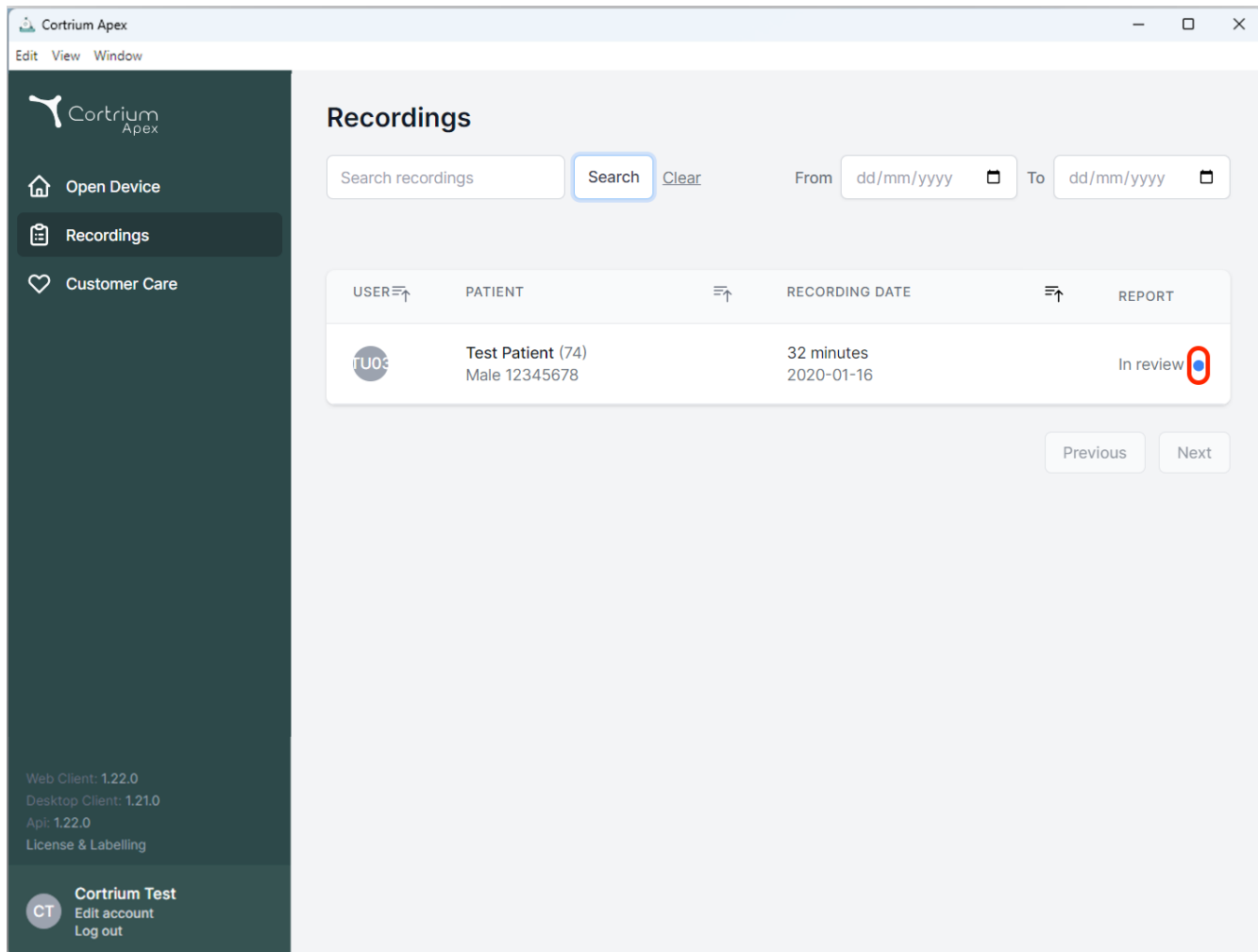
Recordings

Search recordings [Clear](#) From To


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●

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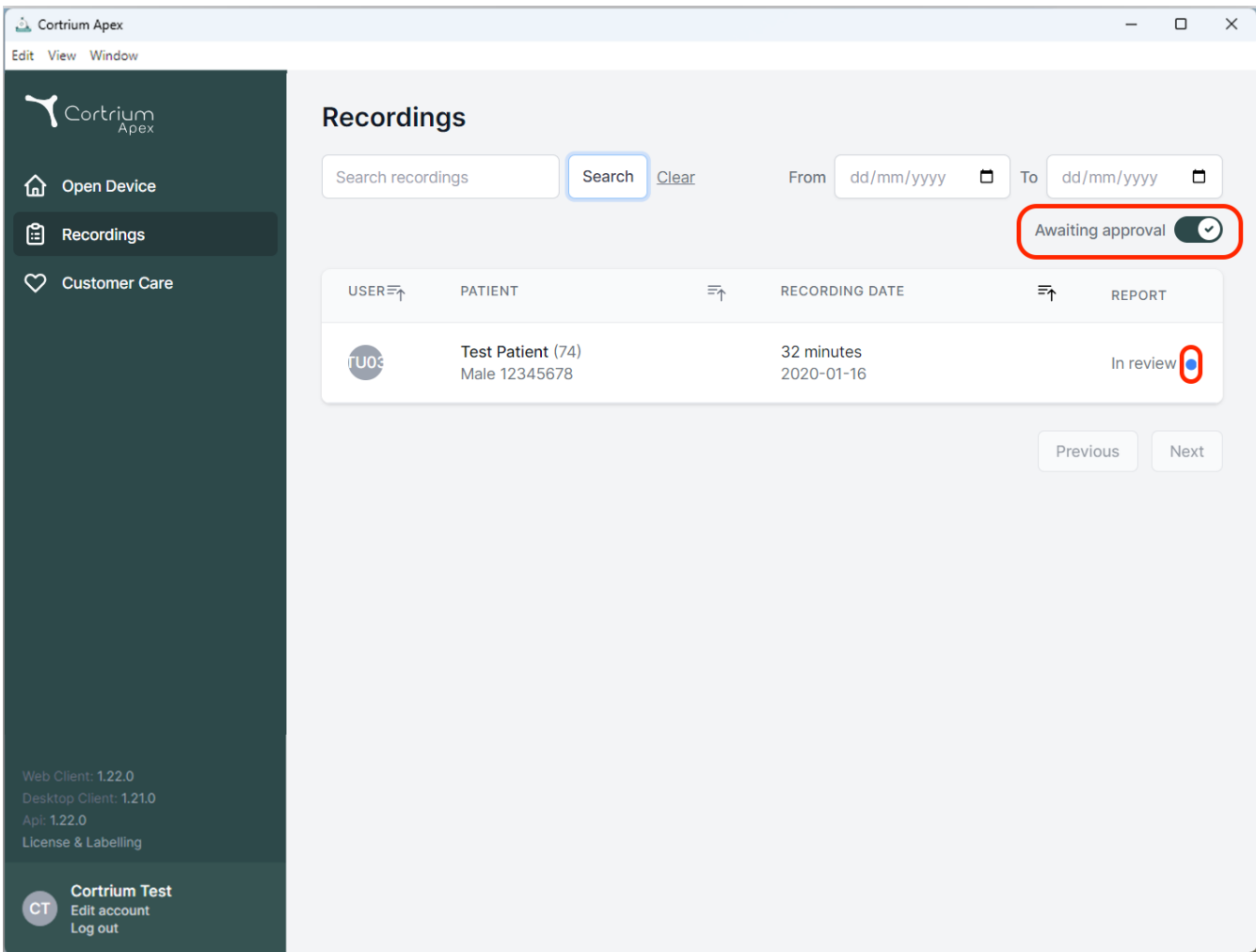
From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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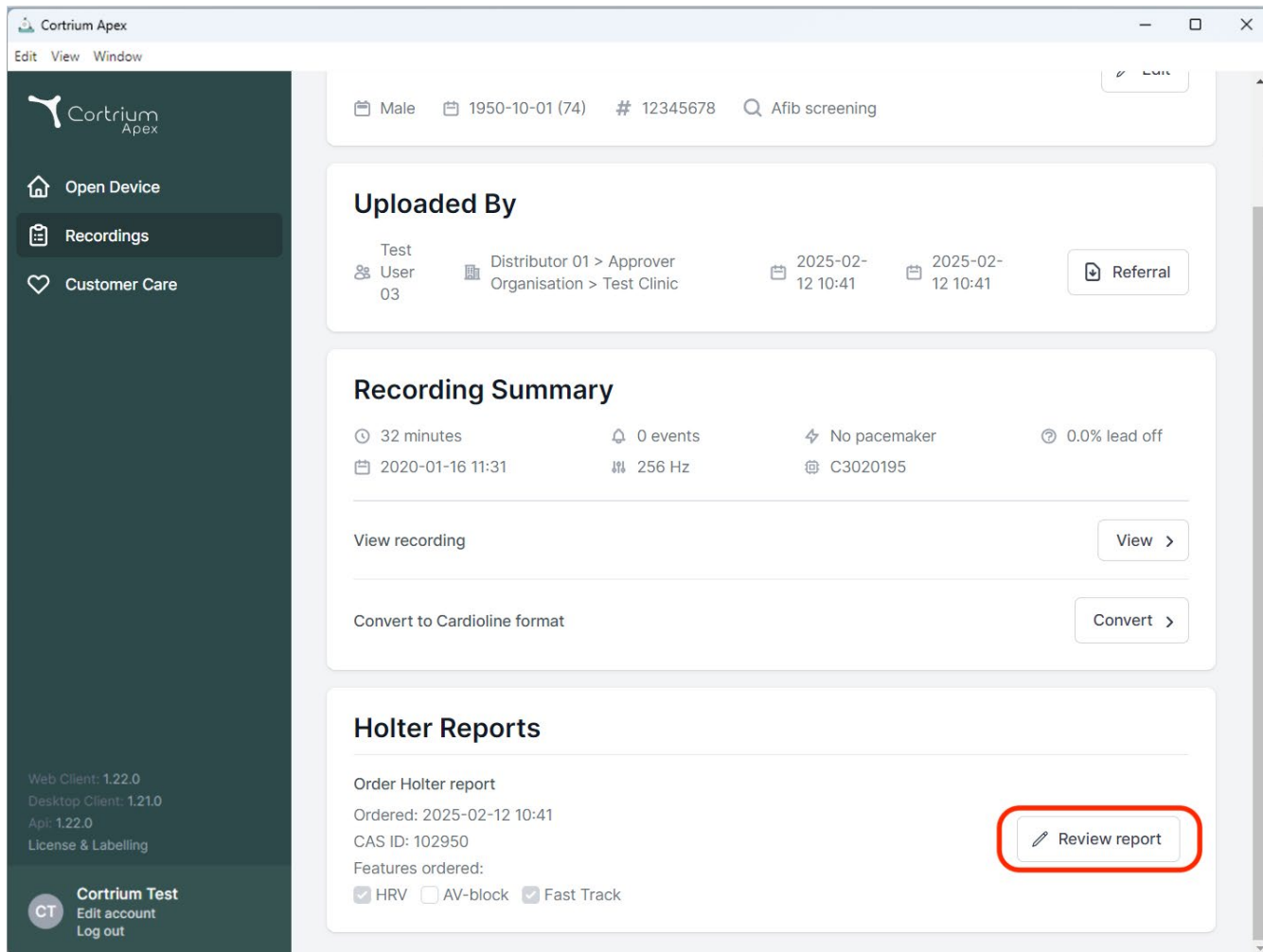The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and a 'Referral' button. The 'Recording Summary' section displays '32 minutes', '0 events', 'No pacemaker', '0.0% lead off', '2020-01-16 11:31', '256 Hz', and 'C3020195'. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

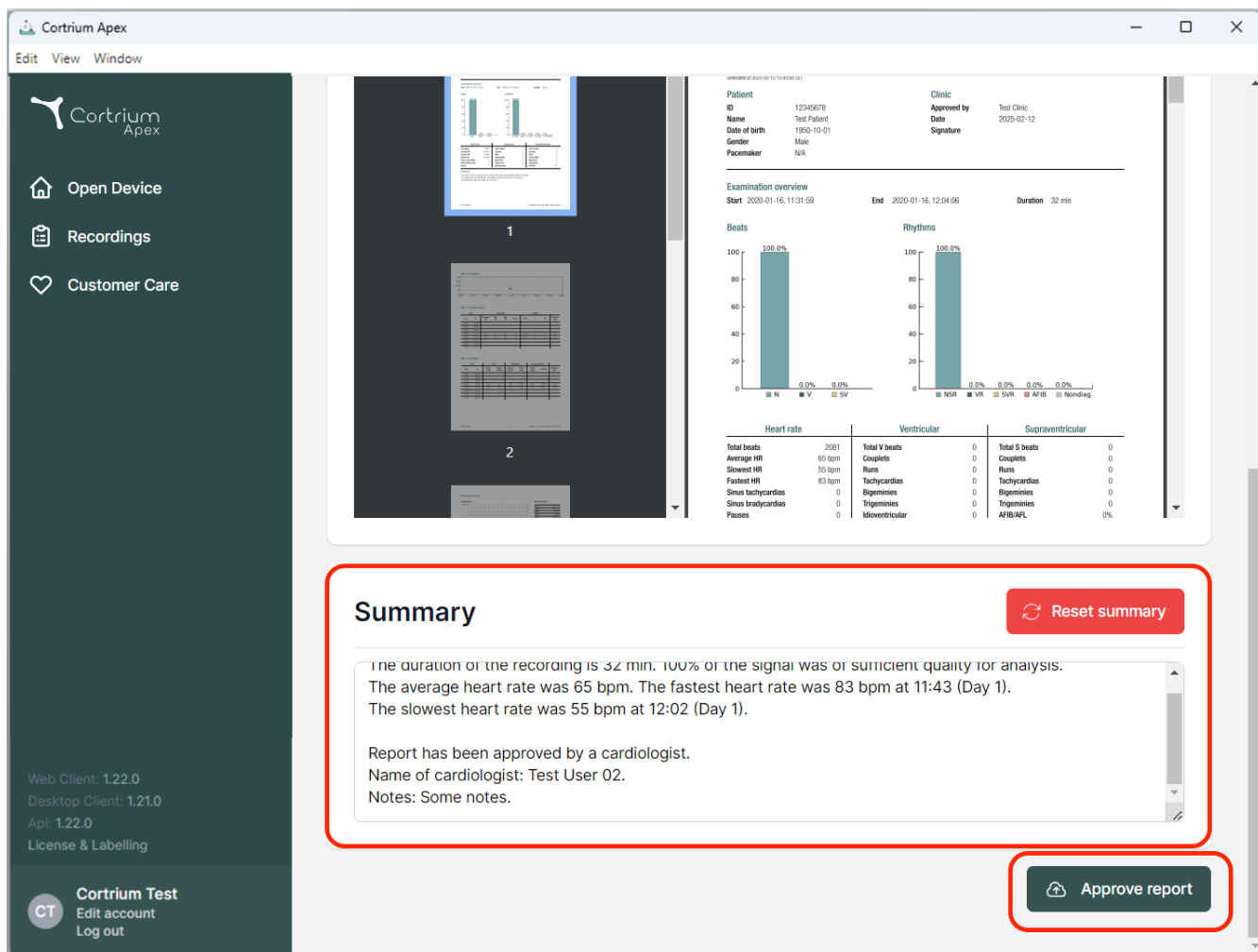
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area shows a patient's Holter monitoring report. At the top, there's a patient information section with fields for ID, Name, Date of birth, Gender, and Pacemaker. Below this is an 'Examination overview' section with a table showing Start, End, and Duration. The report includes two bar charts: 'Beats' and 'Rhythms'. The 'Beats' chart shows 100.0% for NS (Normal Sinus) and 0.0% for V (Ventricular), SV (Supraventricular), and AF (Atrial Fibrillation). The 'Rhythms' chart shows 100.0% for NSR (Normal Sinus Rhythm) and 0.0% for VR (Ventricular Rhythm), SVR (Supraventricular Rhythm), AFB (Atrial Fibrillation), and Nonrhythm. Below the charts is a table with three columns: Heart rate, Ventricular, and Supraventricular. The 'Heart rate' column lists Total beats (2081), Average HR (65 bpm), Slowest HR (55 bpm), Fastest HR (83 bpm), Sinus tachycardia (0), Sinus bradycardia (0), and Pauses (0). The 'Ventricular' column lists Total V beats (0), Couplets (0), Runs (0), Tachycardia (0), Bigeminy (0), Trigeminy (0), and Idioventricular (0). The 'Supraventricular' column lists Total S beats (0), Couplets (0), Runs (0), Tachycardia (0), Bigeminy (0), Trigeminy (0), AFB/ATL (0%), and Nonrhythm (0%).

Summary Reset summary

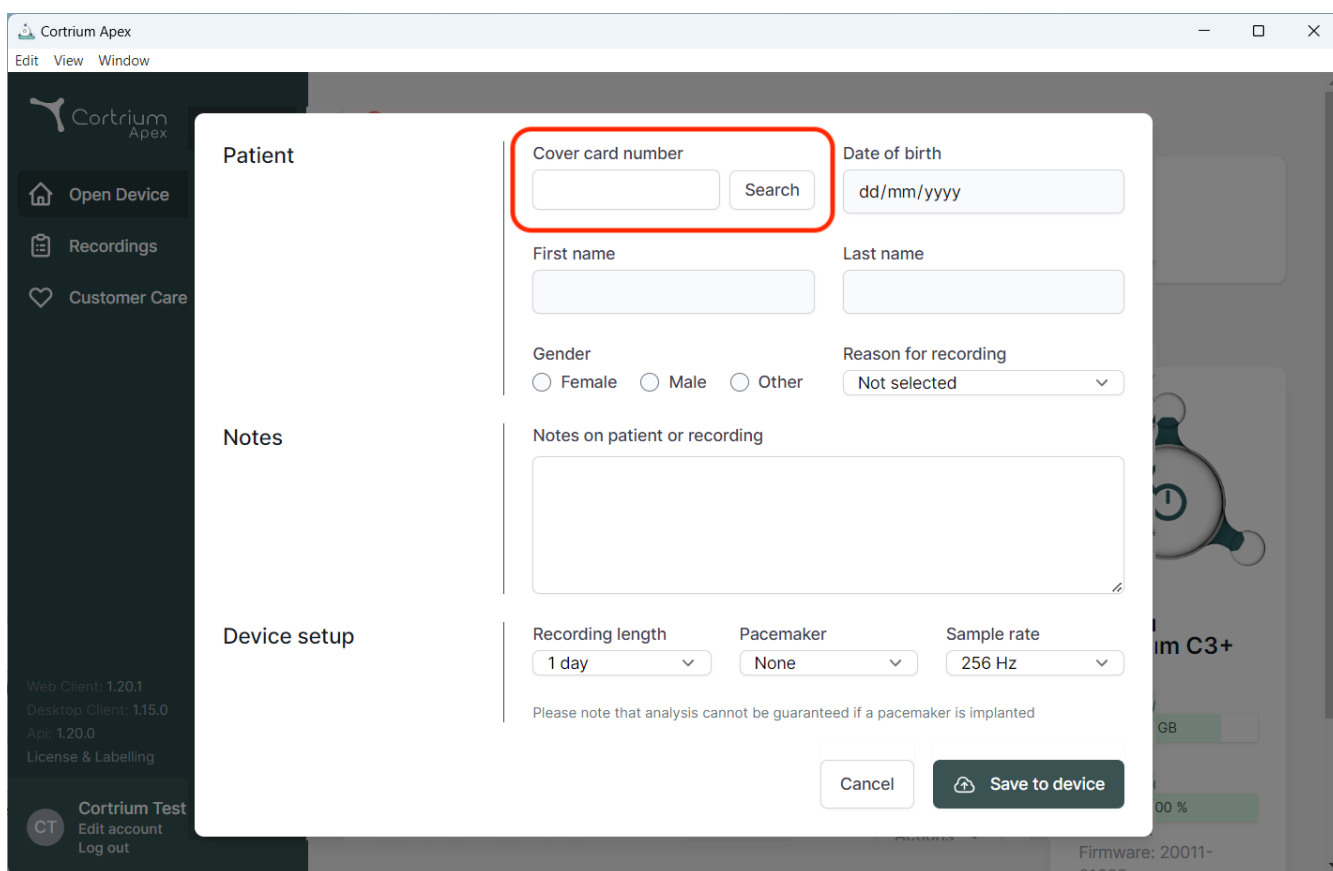
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient search and recording configuration. The modal is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test" with options to "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dania
E-mail: info@cortrium.com • Strona internetowa: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47