Cortrium Apex 1.26

Cortrium Apex 1.26

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
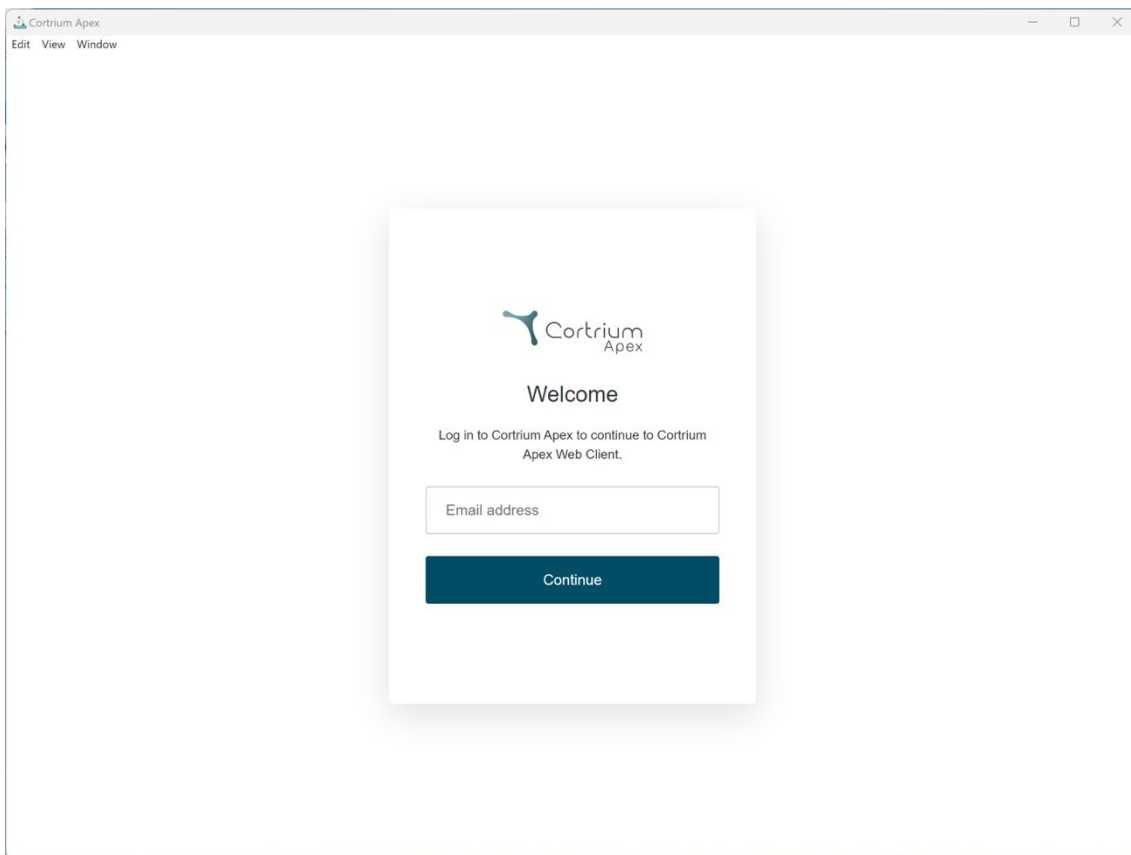
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle drop shadow. At the top of the form is the Cortrium Apex logo. Below the logo is the heading "Welcome". Underneath the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." followed by a text input field labeled "Email address". At the bottom of the form is a dark blue button with the text "Continue" in white.

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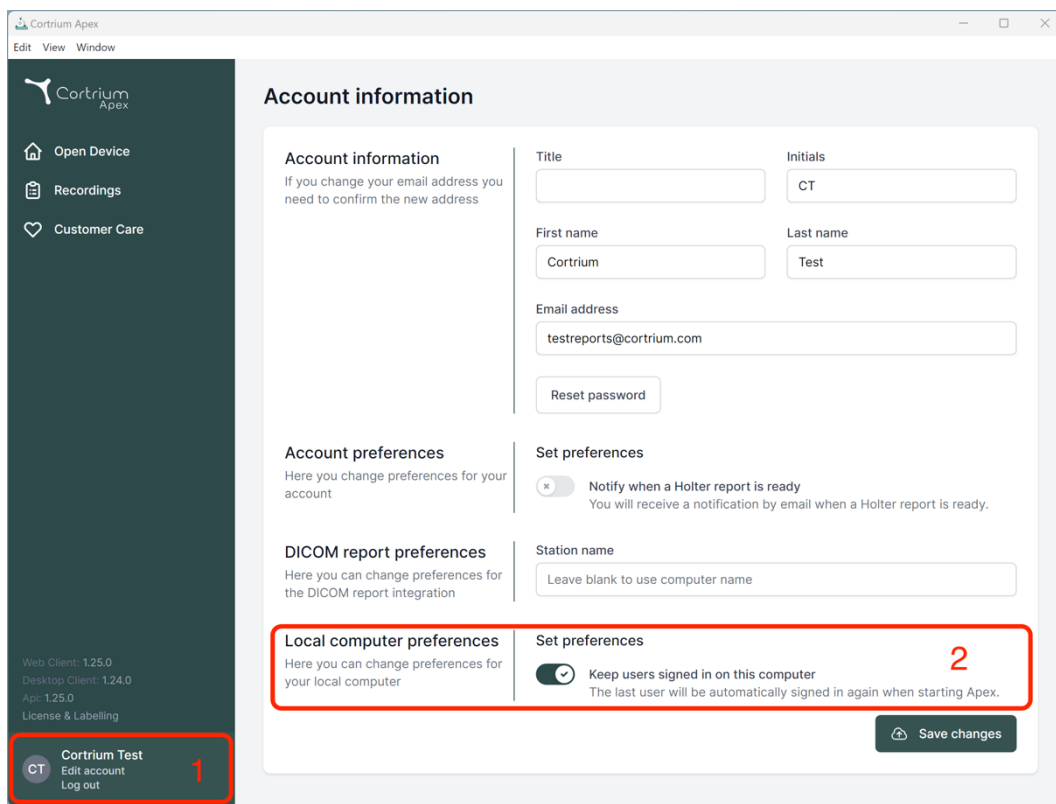The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0) and a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section includes a 'Set preferences' area with a toggle for 'Keep users signed in on this computer'. Red boxes and numbers highlight the 'Cortrium Test' user profile in the sidebar (labeled 1) and the 'Local computer preferences' section (labeled 2).

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[Reset password](#)

Account preferences

Here you can change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

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Station name

Local computer preferences

Here you can change preferences for your local computer

Set preferences

☒ Keep users signed in on this computer
The last user will be automatically signed in again when starting Apex.

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
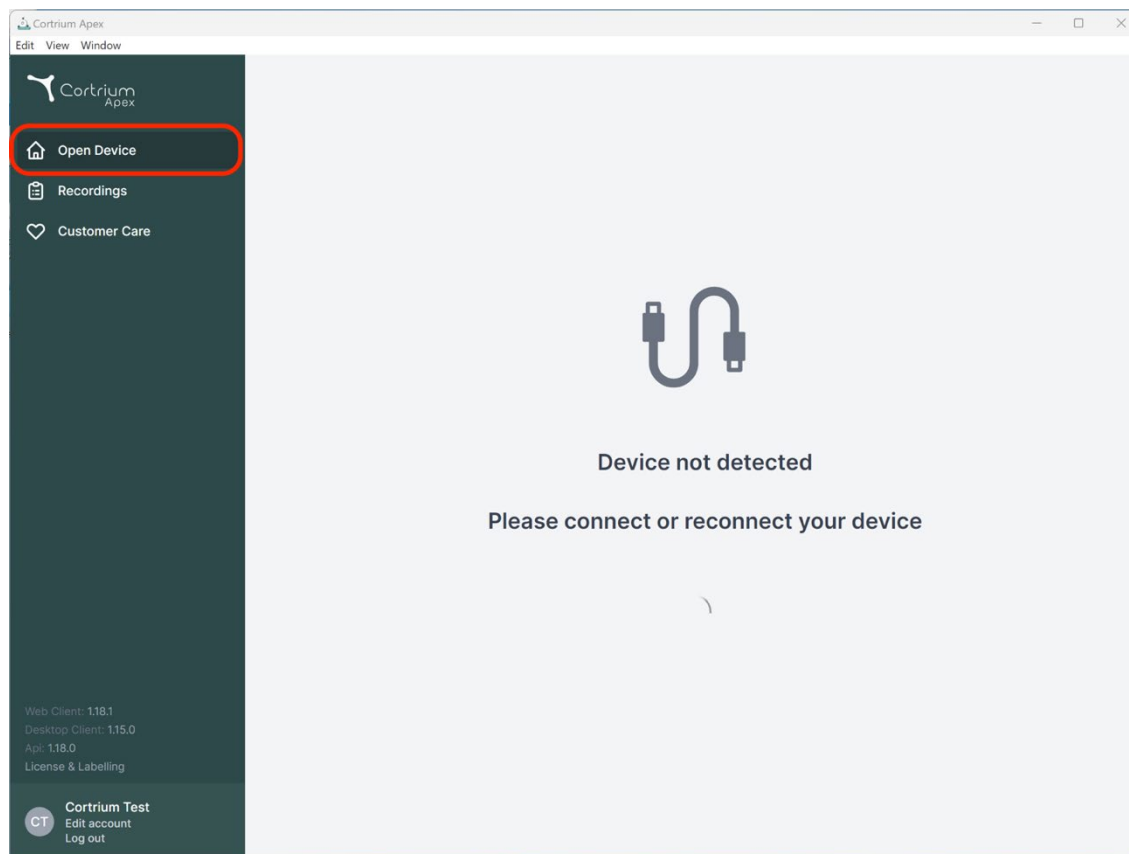
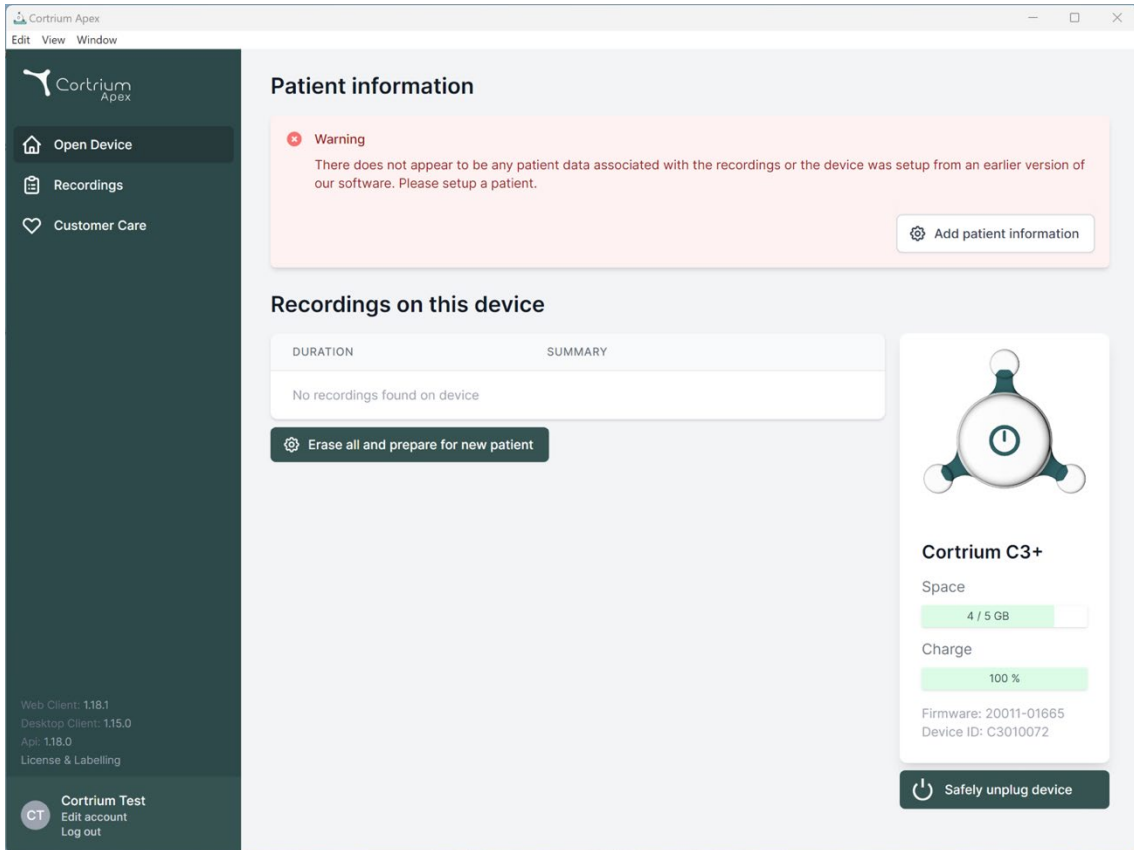
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' displays 'Space' (4 / 5 GB) and 'Charge' (100 %) with progress bars, and provides 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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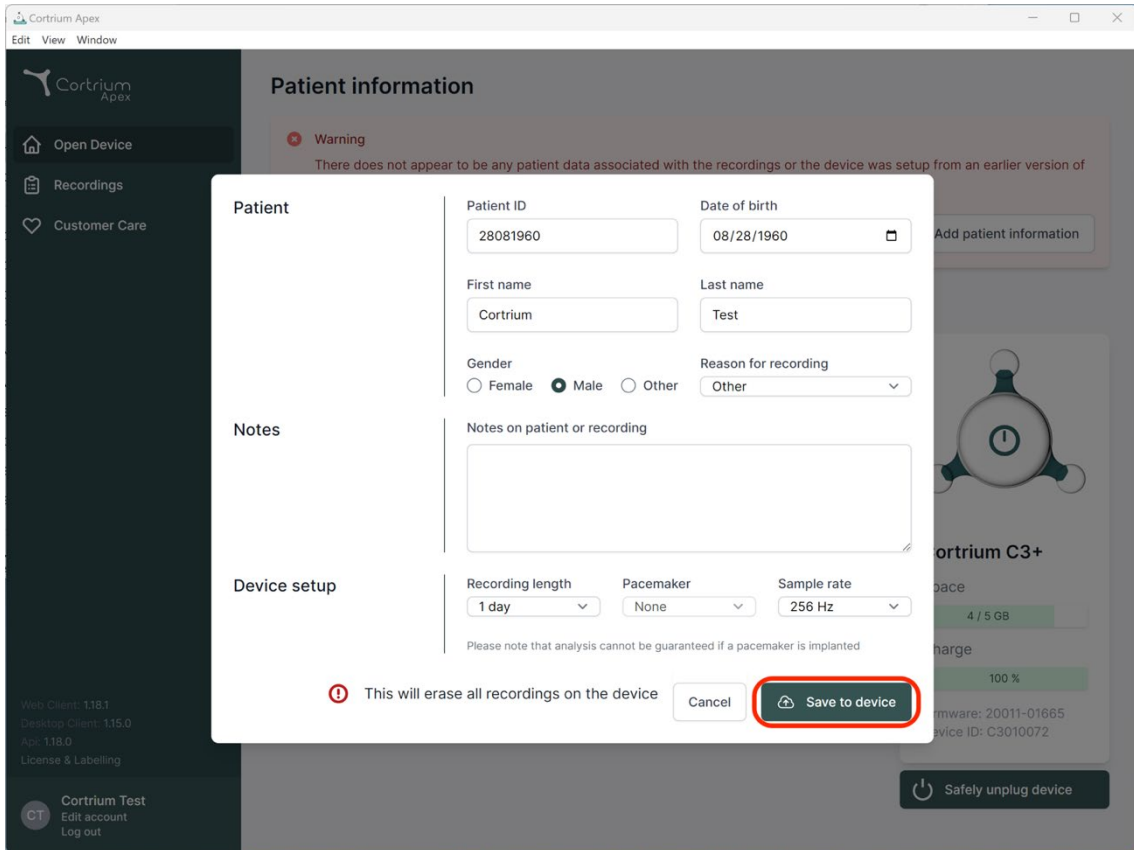Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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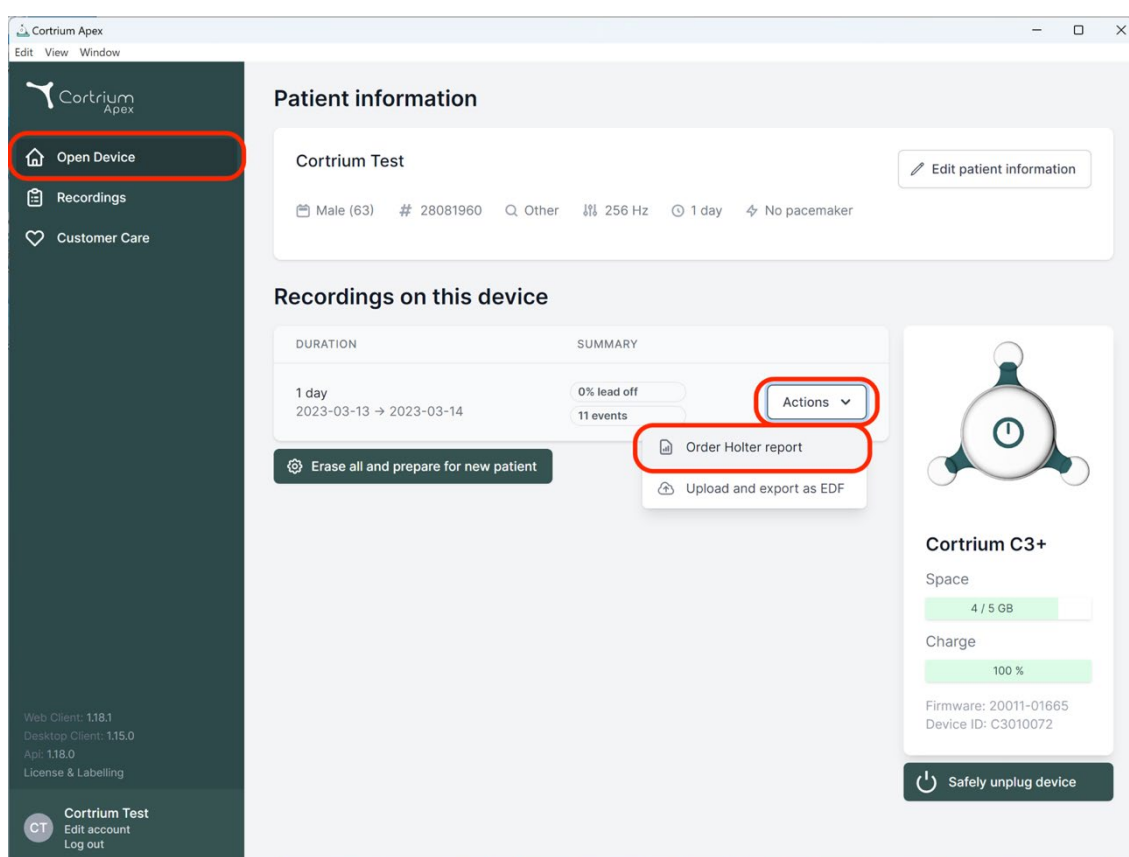
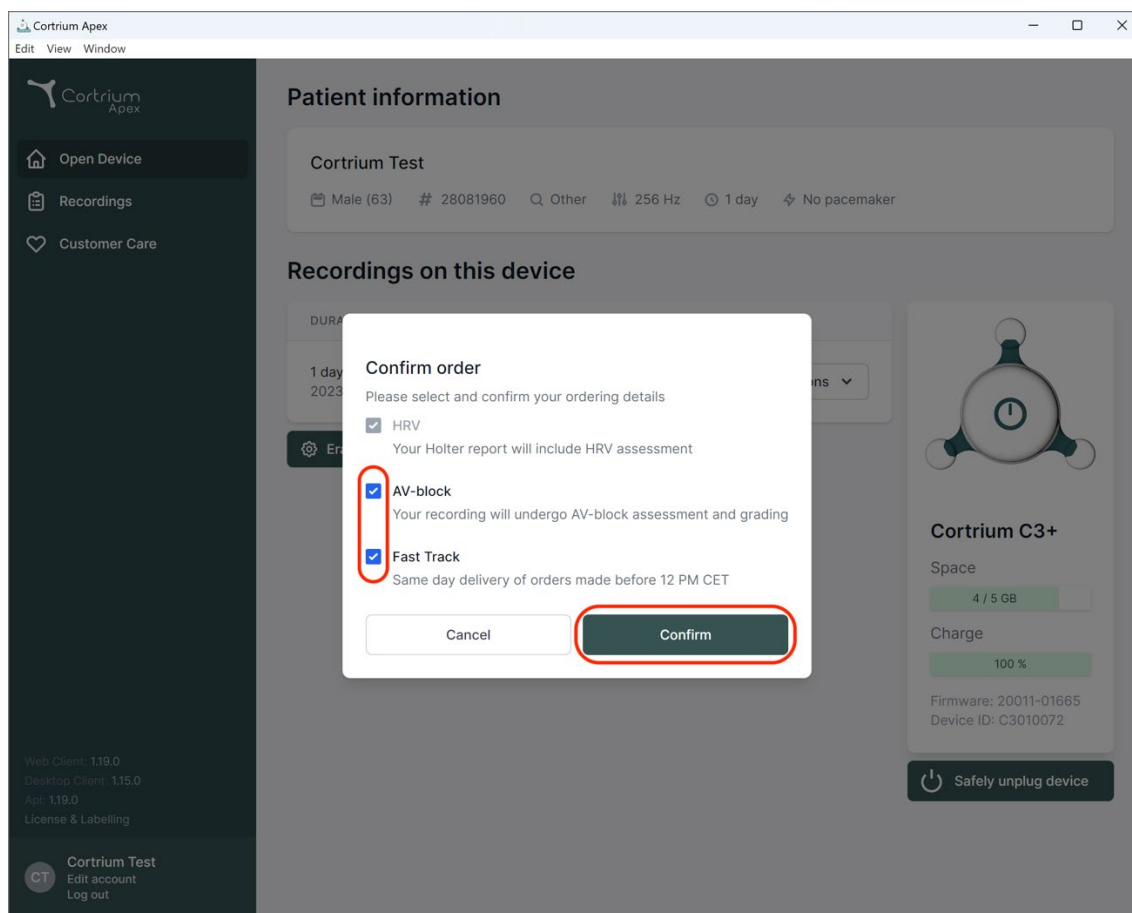
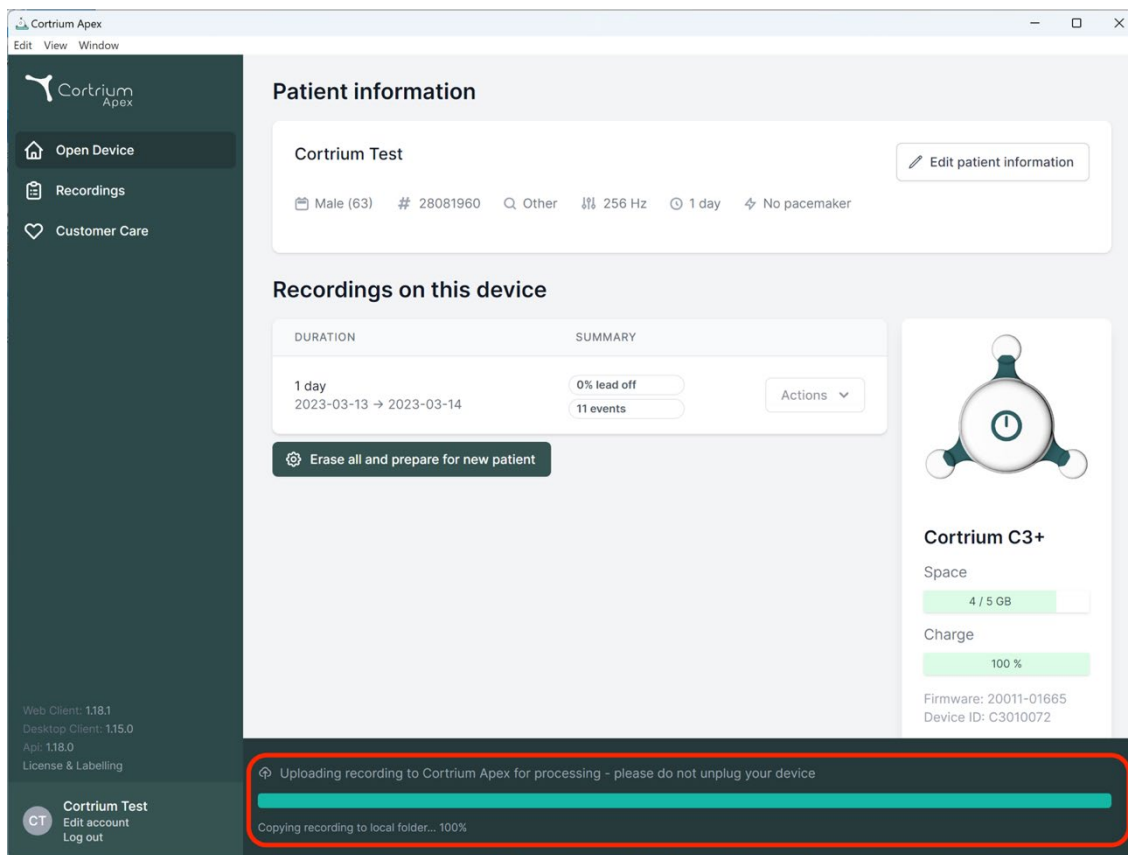
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and device settings. Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom of the interface, which contains the text: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. To the right of the recording table, there is a device status card for 'Cortrium C3+' showing space usage (4 / 5 GB) and charge level (100%).

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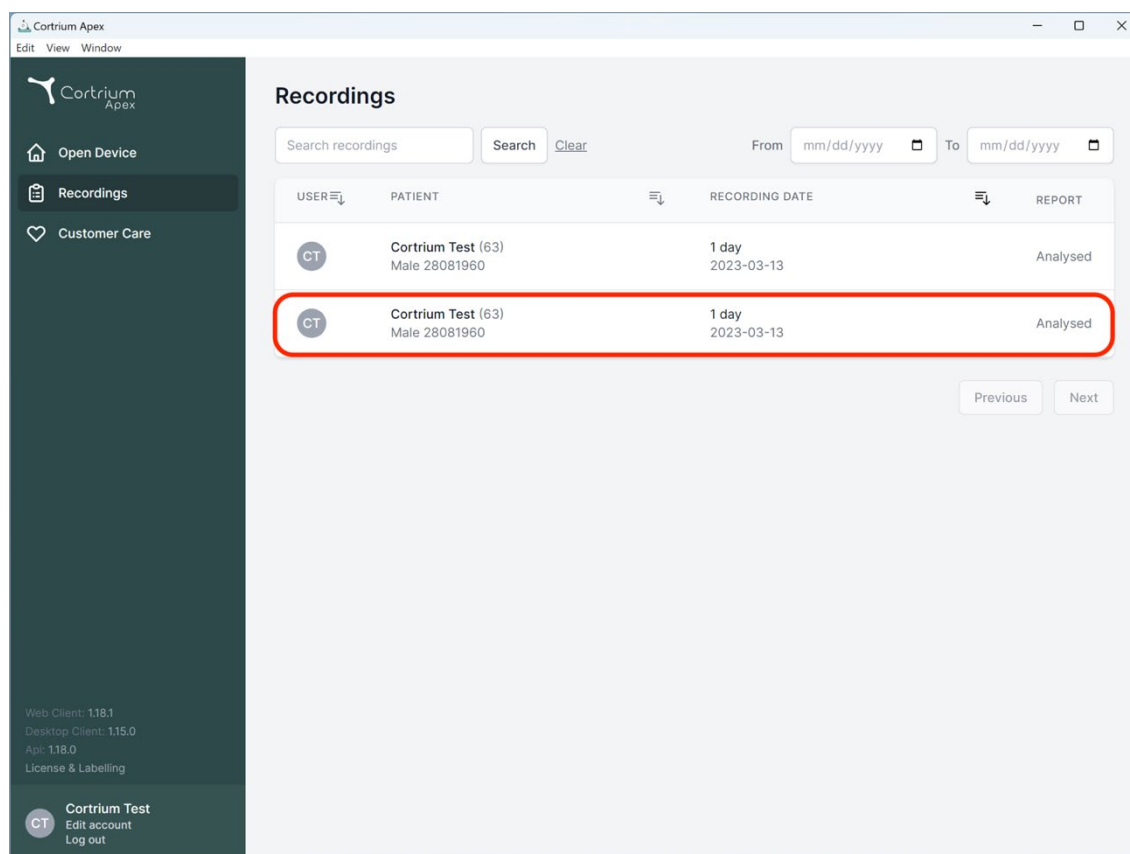
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
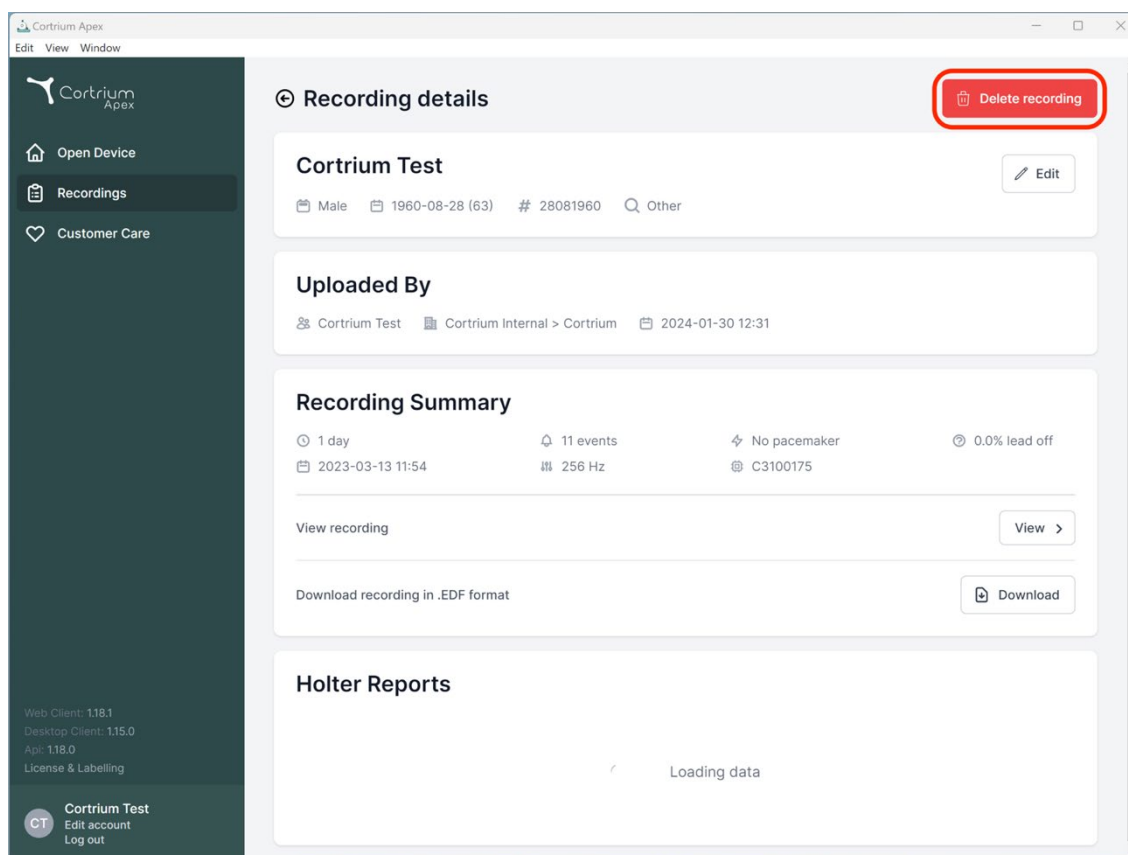
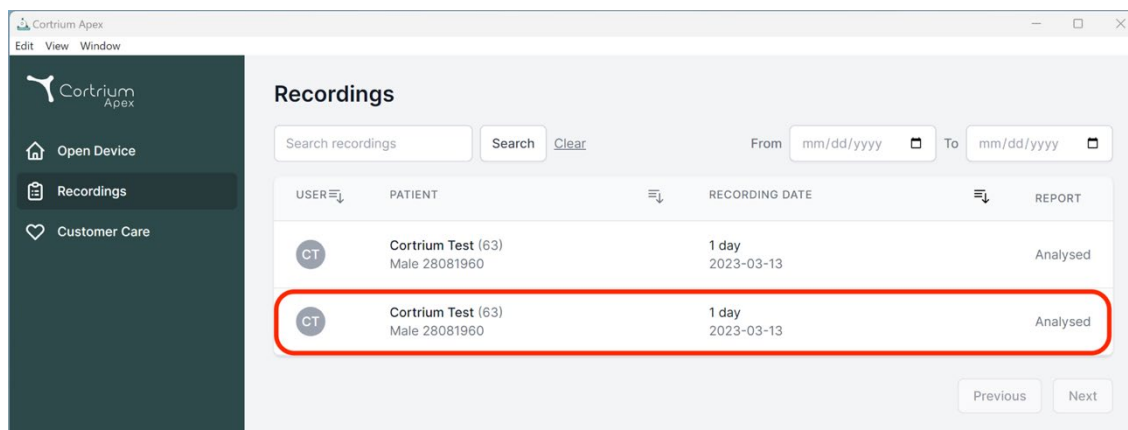
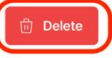
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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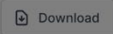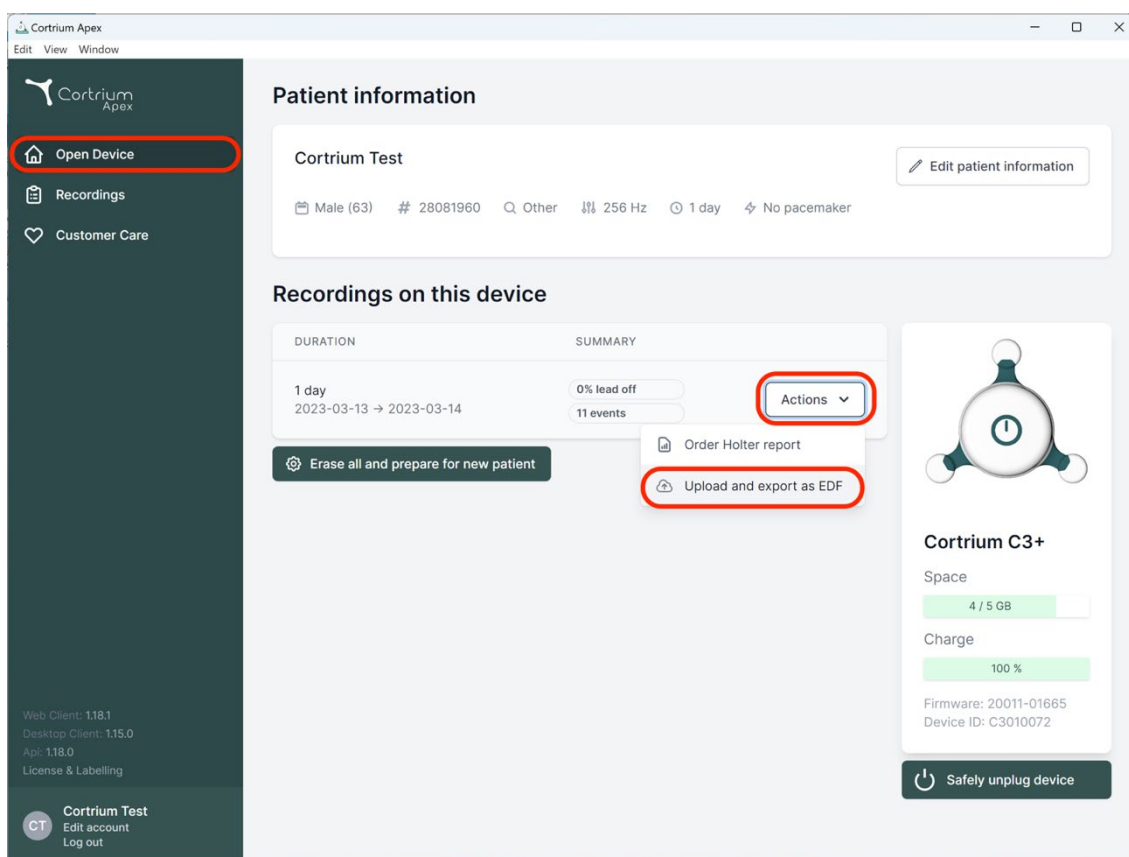
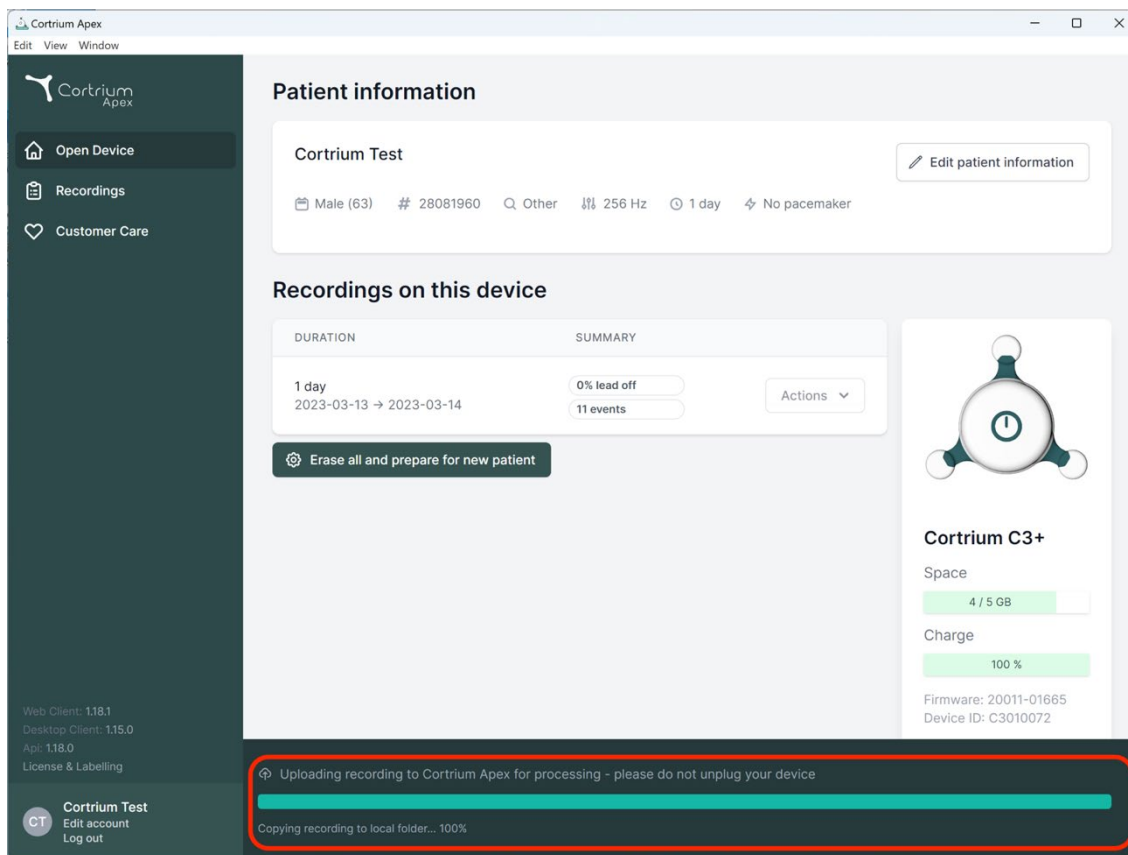
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

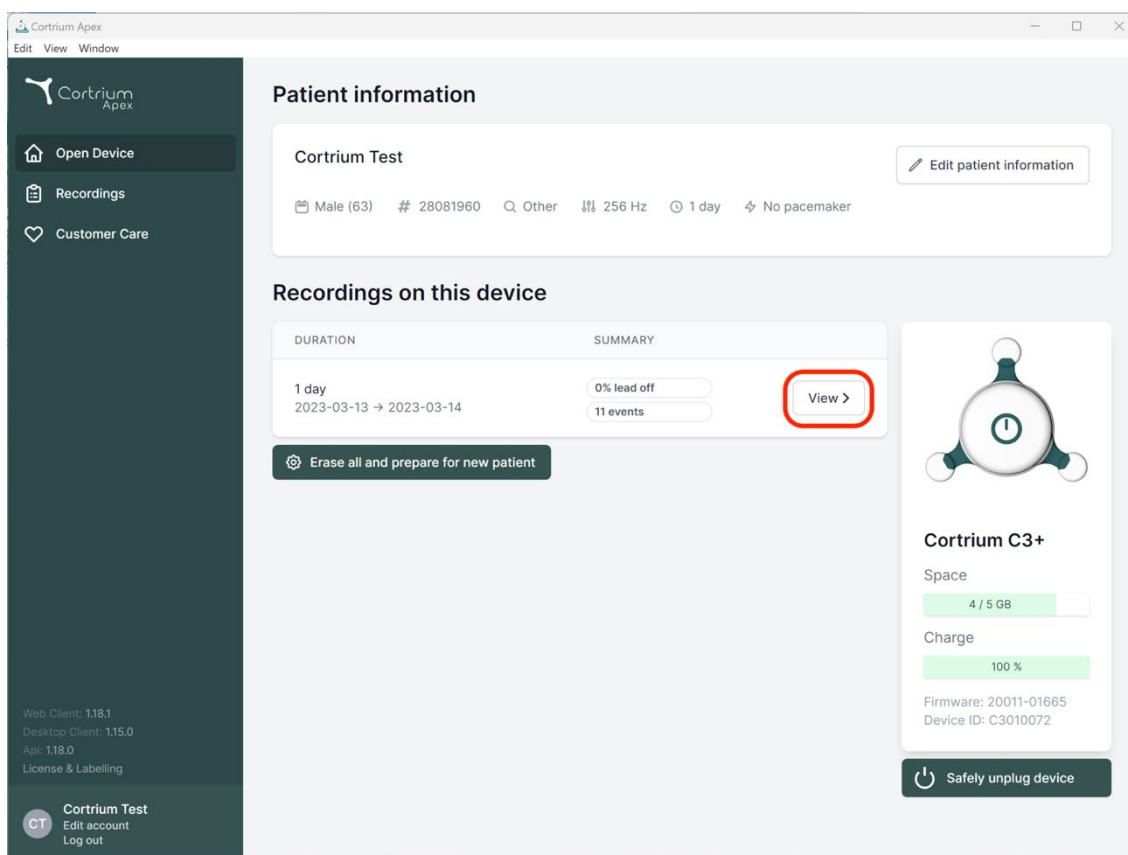
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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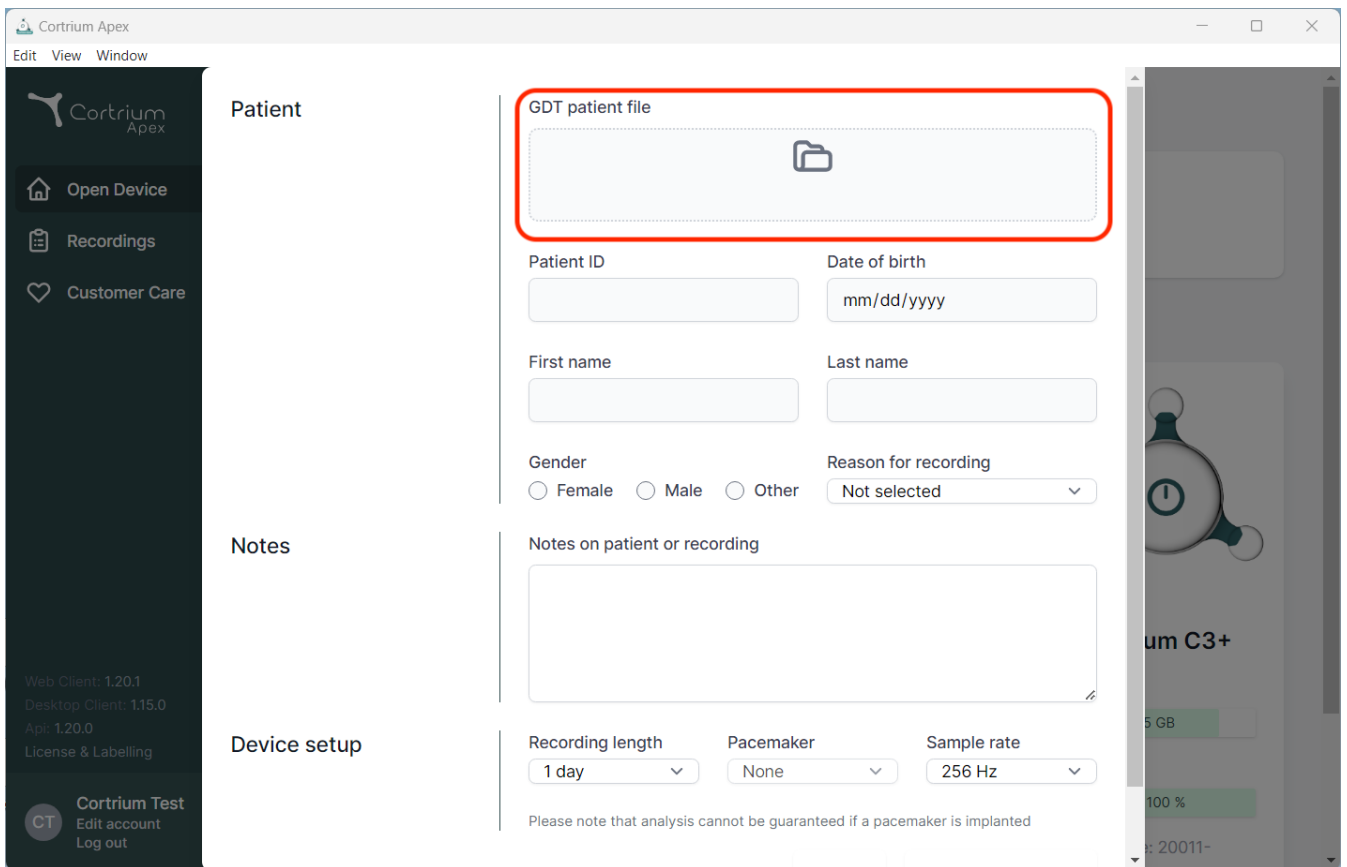
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

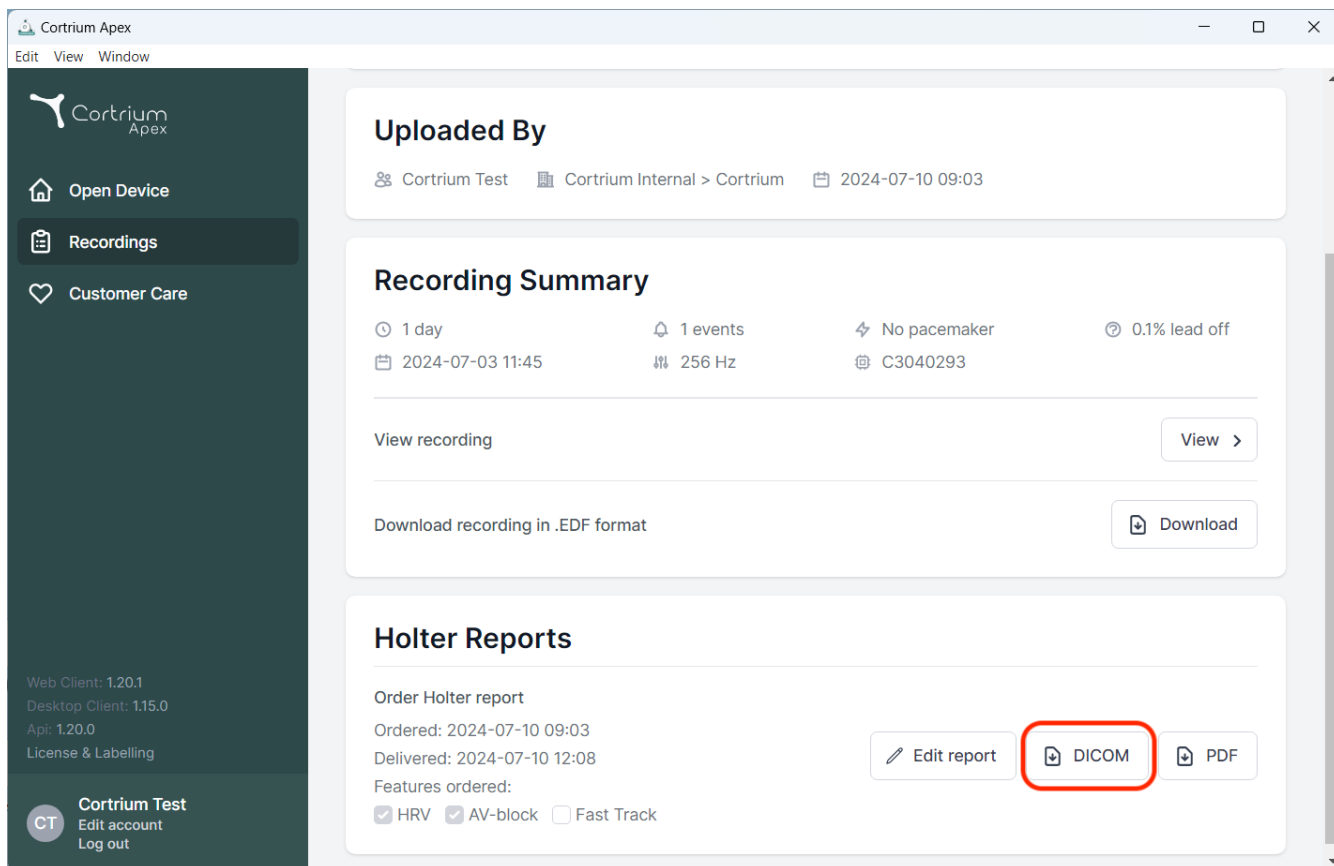
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section features a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a "Cortrium C3+" device with a battery level indicator at 100% and a date "2011-".

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying recording details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'. Below these buttons, the 'Features ordered' section shows 'HRV' and 'AV-block' as selected options, and 'Fast Track' as an unselected option.

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Account information

Account information

If you change your email address you need to confirm the new address

Title

Initials

First name

Last name

Email address

Reset password

Account preferences

Here you change preferences for your account

Set preferences

☒

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

DICOM report preferences

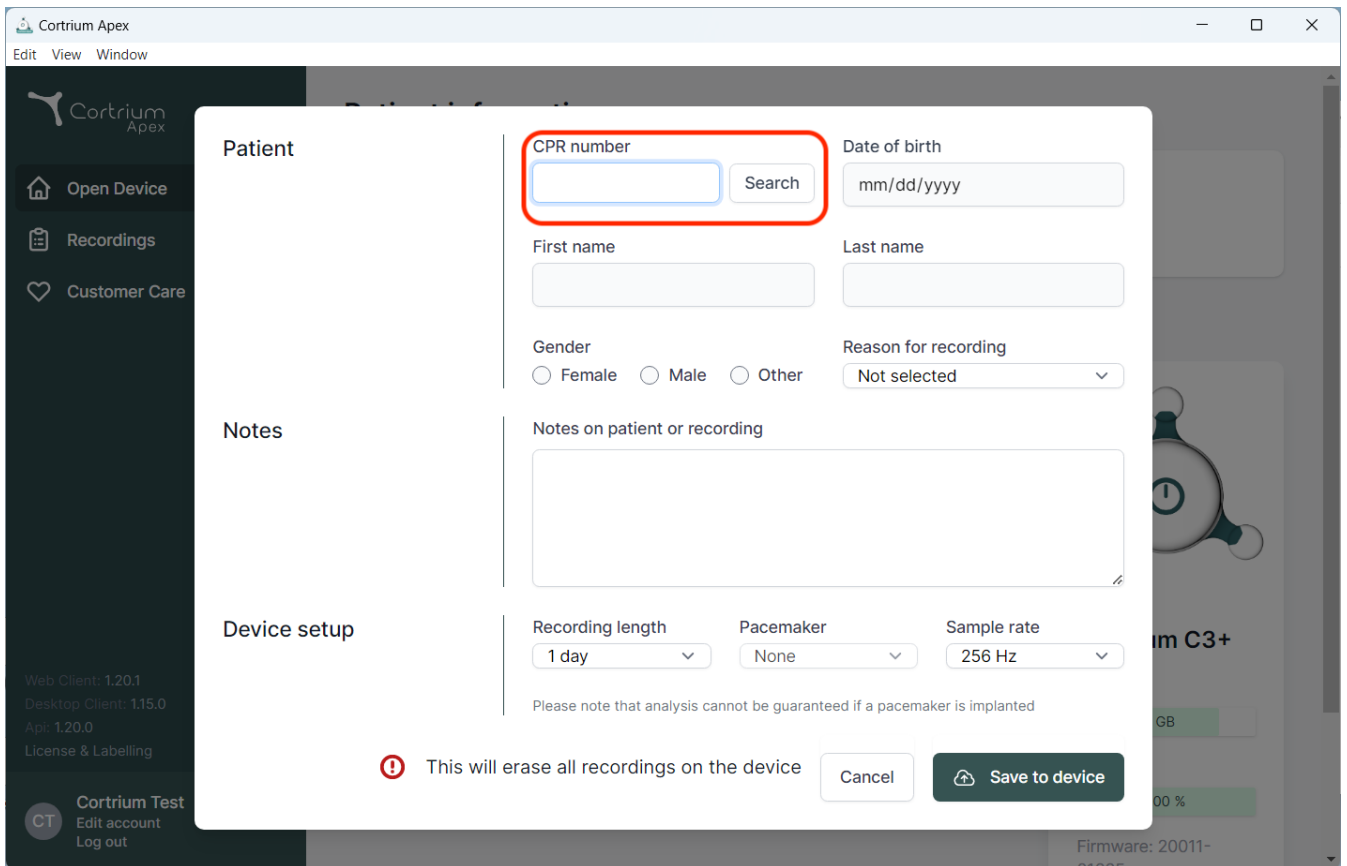
Here you can change preferences for the DICOM report integration

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
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient search and data entry. The form includes fields for "CPR number" (highlighted with a red box), "Date of birth", "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu). There is also a "Notes on patient or recording" text area. Below these fields, the "Device setup" section includes "Recording length" (set to 1 day), "Pacemaker" (set to None), and "Sample rate" (set to 256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". At the bottom right of the modal, there are "Cancel" and "Save to device" buttons. The background shows the main application menu with options like "Open Device", "Recordings", and "Customer Care".

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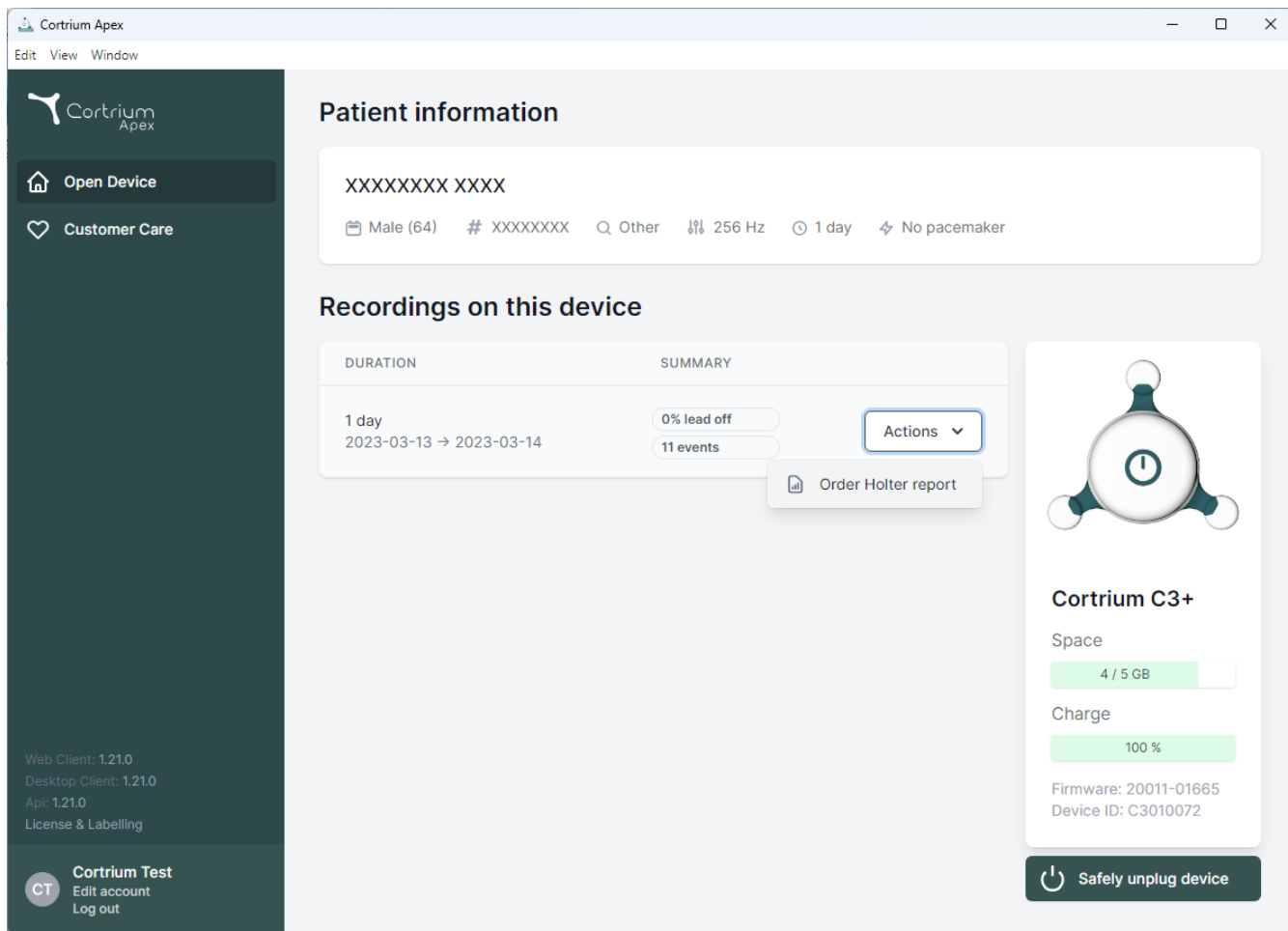The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions

[Order Holter report](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

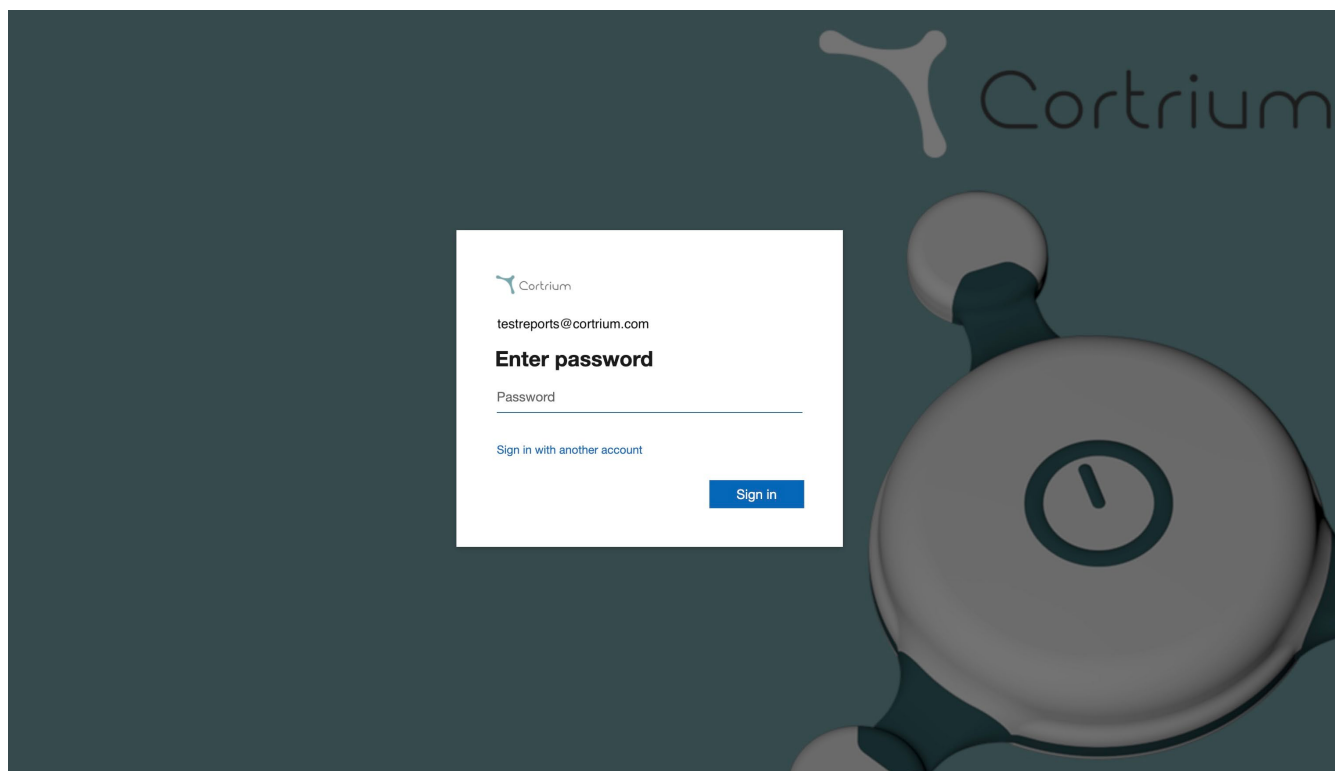
[Safely unplug device](#)

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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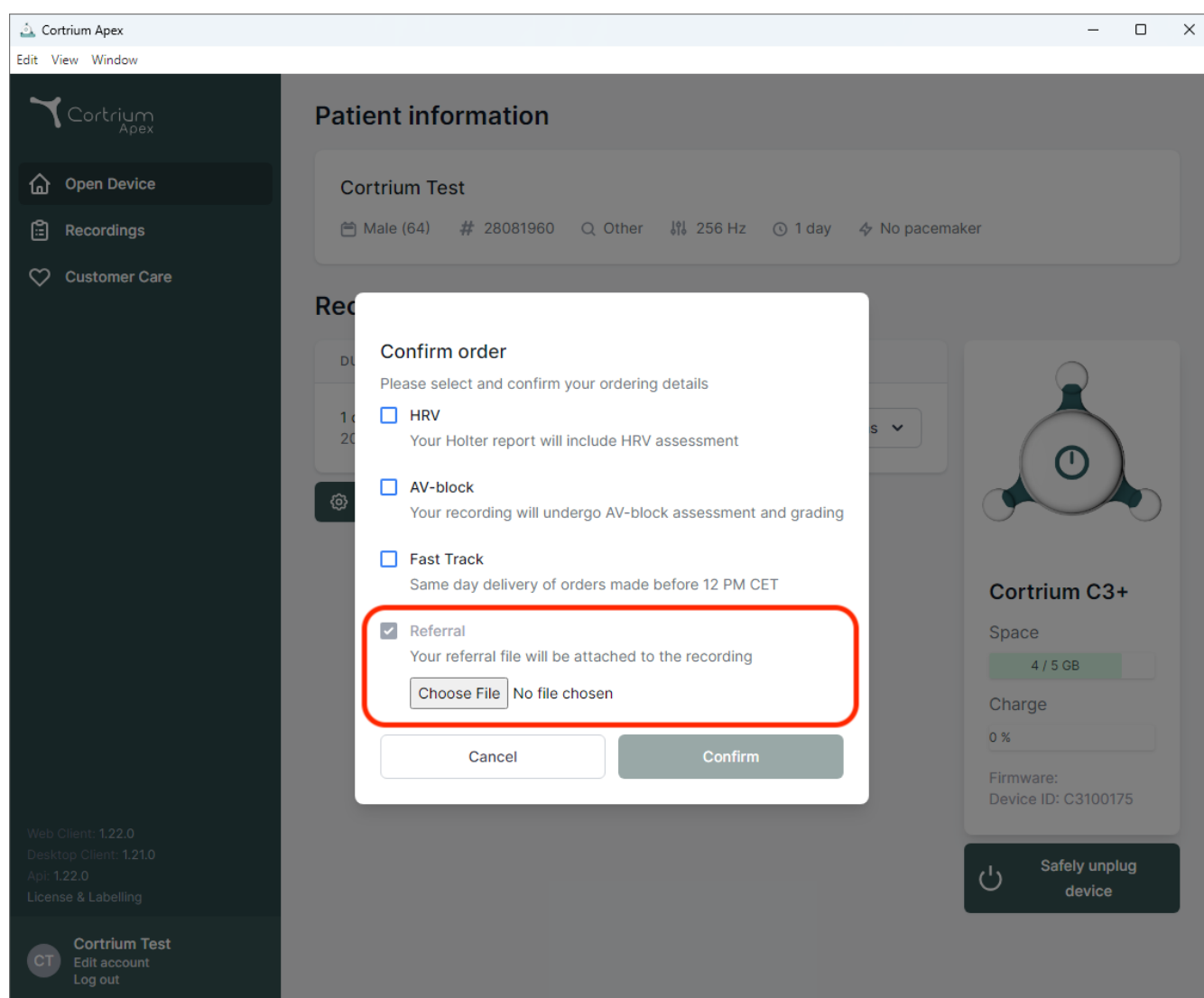
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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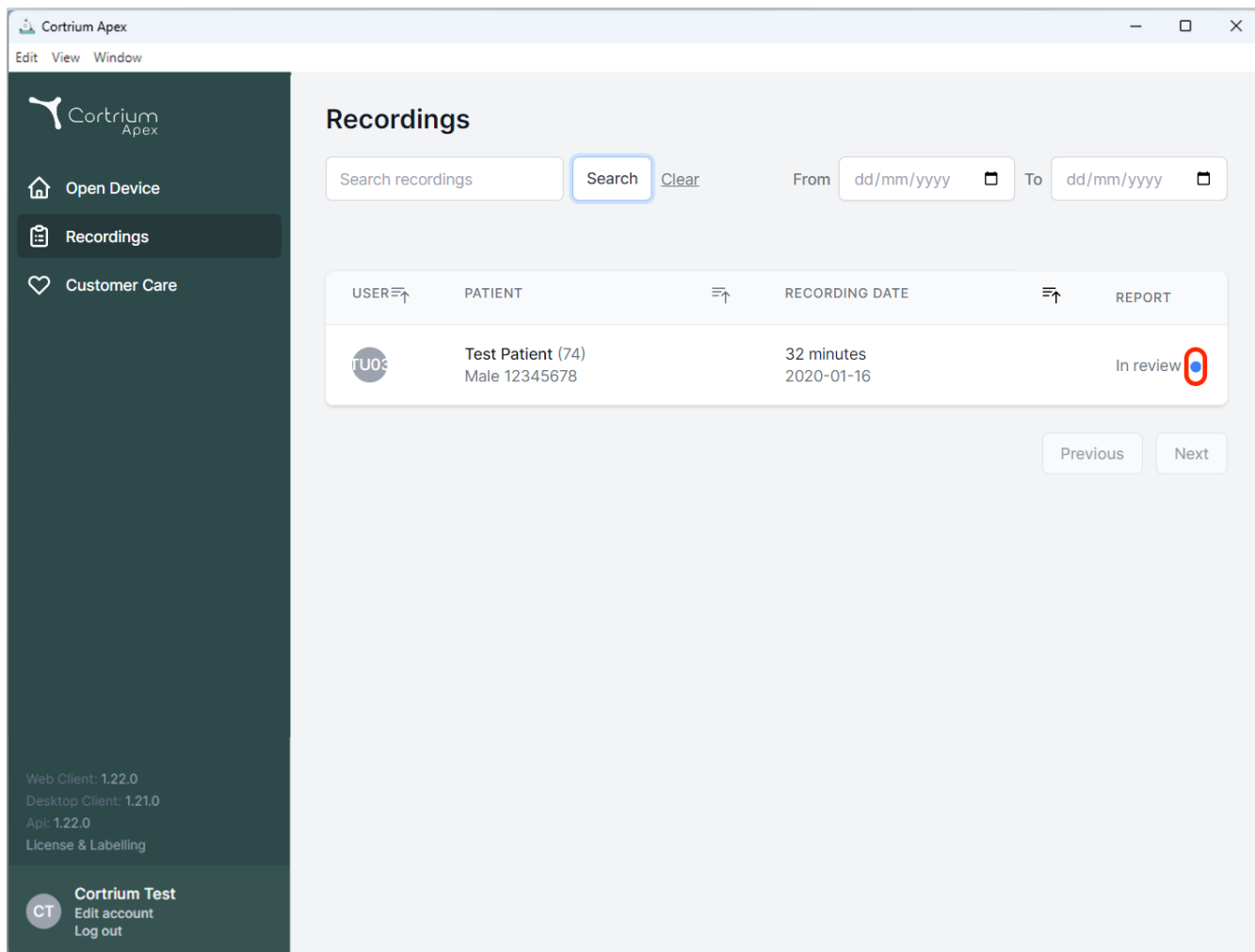
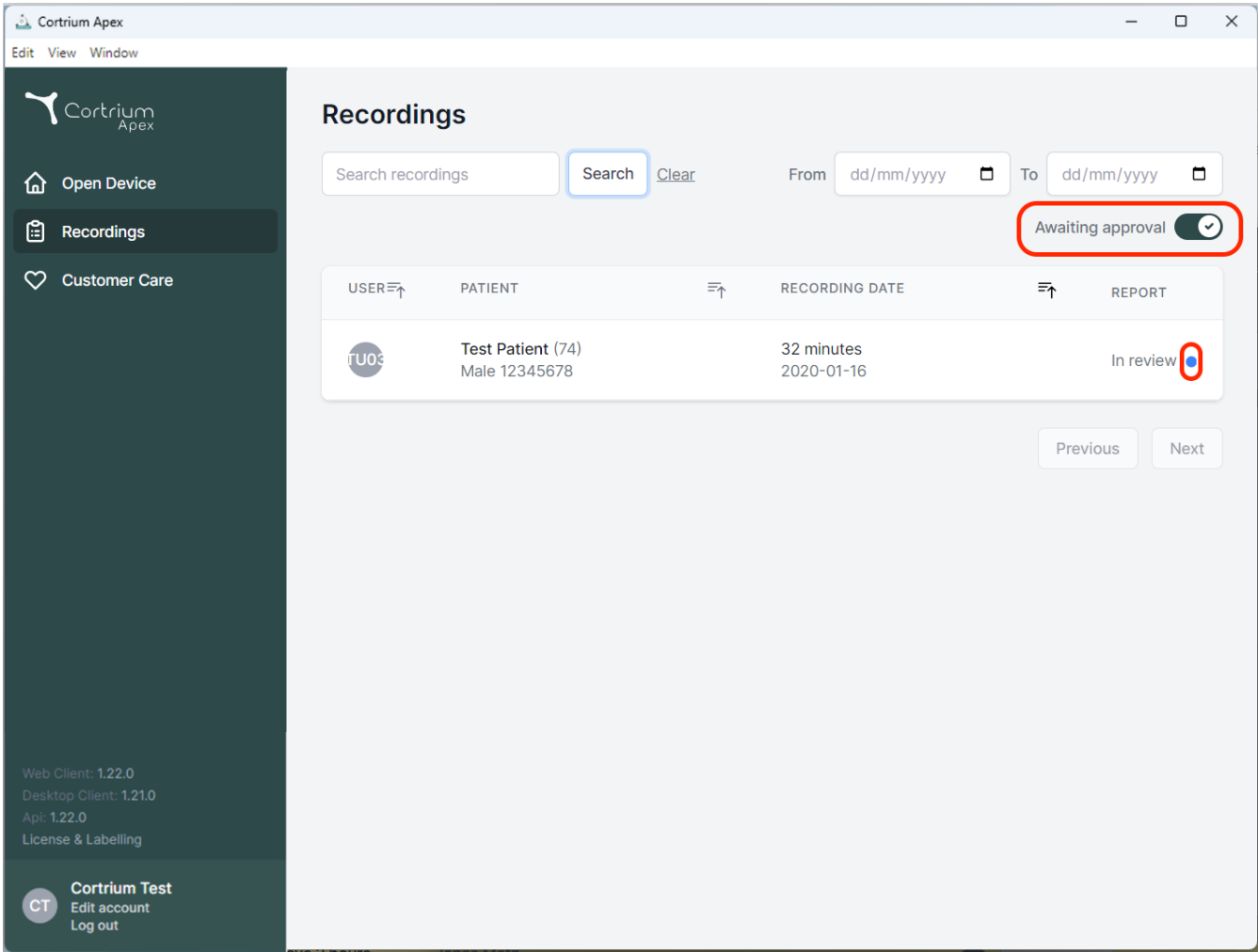
Edit View Window

Recordings

Search recordings Search Clear From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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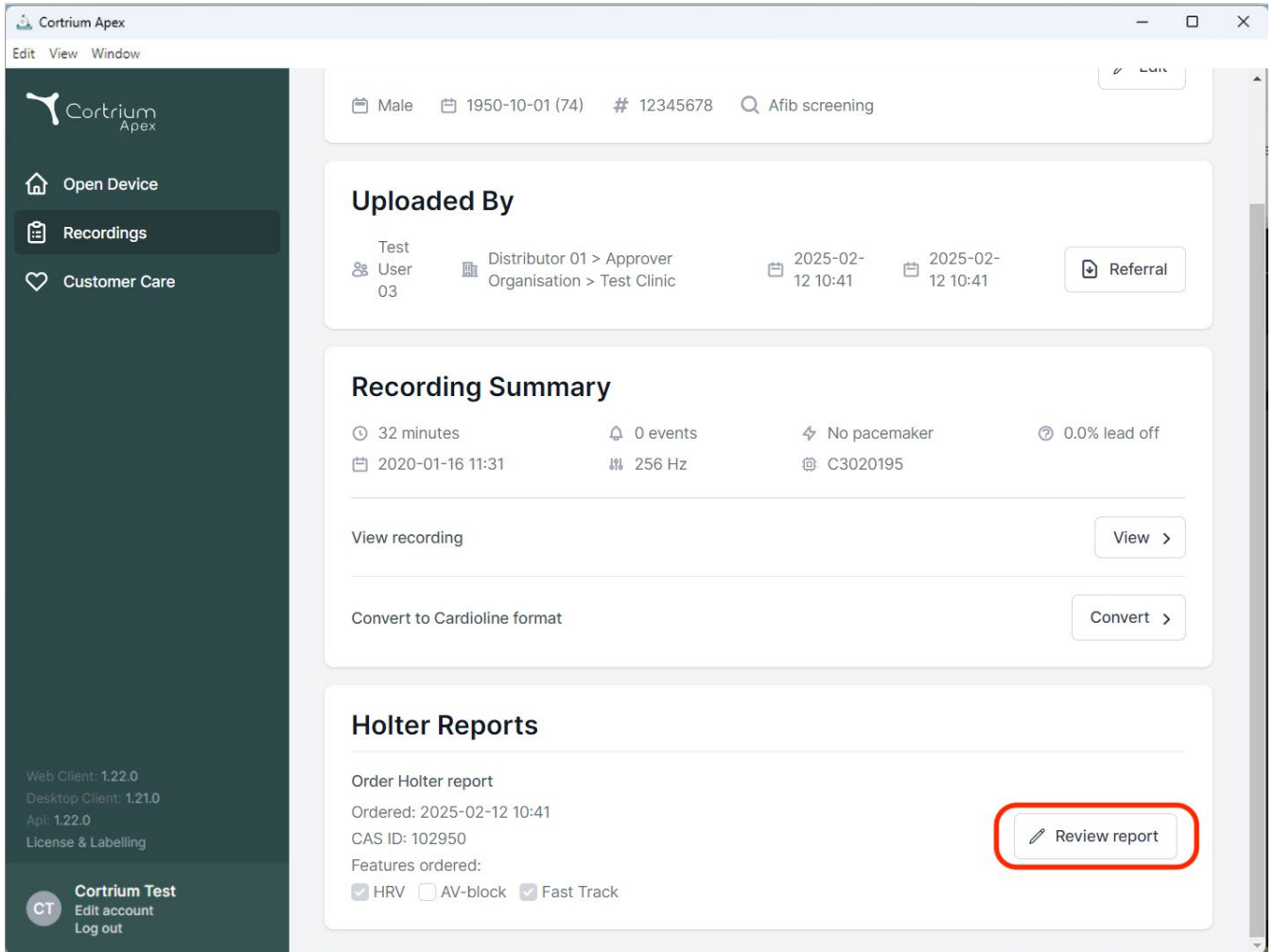
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes "Edit", "View", and "Window" options. The left sidebar contains the Cortrium Apex logo and three main menu items: "Open Device", "Recordings" (highlighted), and "Customer Care". Below the sidebar, version information is listed: "Web Client: 1.22.0", "Desktop Client: 1.21.0", "Apl: 1.22.0", and "License & Labelling". At the bottom of the sidebar, a "Cortrium Test" section includes "Edit account" and "Log out" links.

The main content area displays patient information at the top: "Male", "1950-10-01 (74)", "# 12345678", and "Afib screening". Below this, the "Uploaded By" section shows "Test User 03" as the distributor, with approval details for "Distributor 01 > Approver Organisation > Test Clinic" and timestamps of "2025-02-12 10:41". A "Referral" button is also present.

The "Recording Summary" section provides key metrics: "32 minutes" duration, "0 events", "No pacemaker", and "0.0% lead off". It also includes a date "2020-01-16 11:31", a frequency of "256 Hz", and a device ID "C3020195". Action buttons for "View recording" and "Convert to Cardioline format" are available.

The "Holter Reports" section shows the "Order Holter report" details, including the order date "2025-02-12 10:41" and CAS ID "102950". It lists ordered features: "HRV" (checked), "AV-block" (unchecked), and "Fast Track" (checked). A "Review report" button, highlighted with a red rectangle, is located in the bottom right corner of this section.

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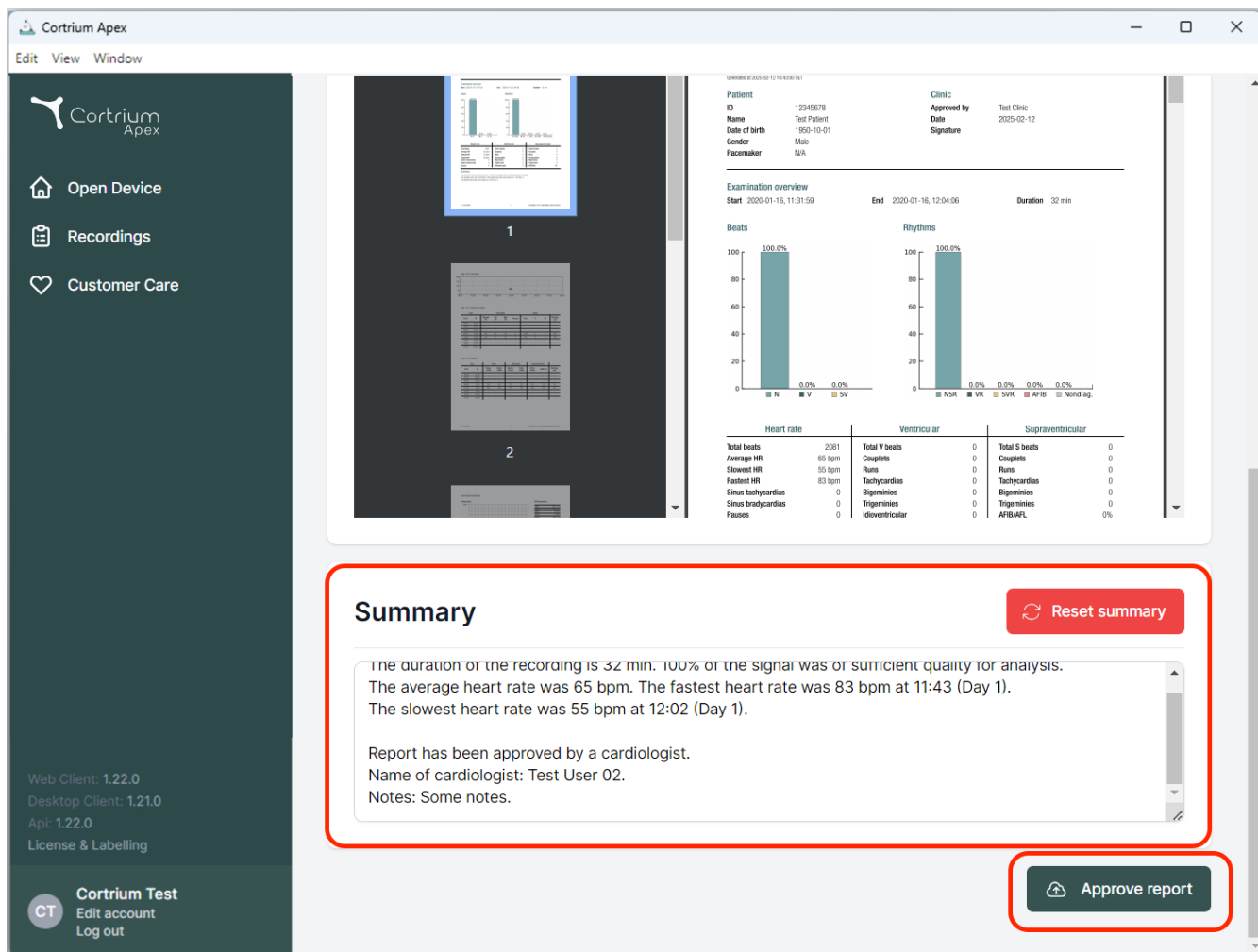The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area shows a patient's Holter report. At the top, there's a patient information section with fields for ID, Name, Date of birth, Gender, and Pacemaker. Below this is an 'Examination overview' section with a timeline from 2020-01-16, 11:31:59 to 2020-01-16, 12:04:06, and a duration of 32 minutes. Two bar charts are shown: 'Beats' (100.0% for N, 0.0% for V, 0.0% for SV) and 'Rhythms' (100.0% for NSR, 0.0% for VR, 0.0% for SVR, 0.0% for AFB, 0.0% for Nonrhythm). Below the charts are three tables: 'Heart rate', 'Ventricular', and 'Supraventricular'. The 'Heart rate' table shows Total beats: 2081, Average HR: 65 bpm, Slowest HR: 55 bpm, Fastest HR: 83 bpm, Sinus tachycardia: 0, Sinus bradycardia: 0, Pauses: 0. The 'Ventricular' table shows Total V beats: 0, Couplets: 0, Runs: 0, Tachycardia: 0, Bigeminy: 0, Trigeminy: 0, Idioventricular: 0. The 'Supraventricular' table shows Total S beats: 0, Couplets: 0, Runs: 0, Tachycardia: 0, Bigeminy: 0, Trigeminy: 0, AFB/ATL: 0%. A red box highlights the 'Summary' section, which contains the following text: 'The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1). Report has been approved by a cardiologist. Name of cardiologist: Test User 02. Notes: Some notes.' To the right of the summary is a 'Reset summary' button. At the bottom right, there is an 'Approve report' button.

Summary [Reset summary](#)

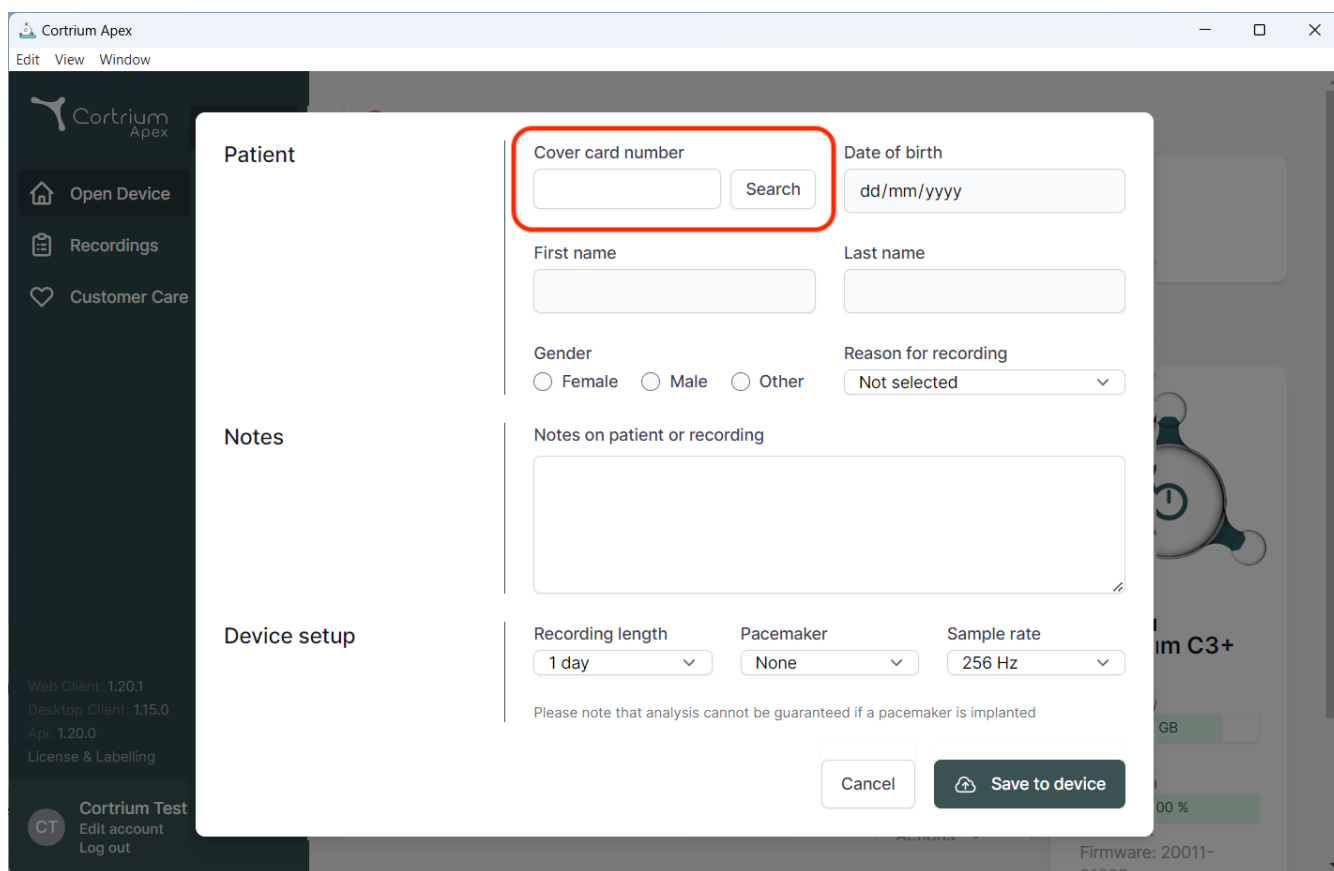
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient information. The form includes the following fields and controls:

- Cover card number:** A text input field, highlighted with a red box, with a "Search" button next to it.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes on patient or recording:** A large text area for notes.
- Device setup:** A section containing:
 - Recording length:** A dropdown menu showing "1 day".
 - Pacemaker:** A dropdown menu showing "None".
 - Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the form, there are two buttons: "Cancel" and "Save to device".

The background shows the Cortrium Apex application menu with options like "Open Device", "Recordings", and "Customer Care". The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test", "Edit account", "Log out", and "Firmware: 20011-01000".

02/2025

Cortrium ApS - Erik Husfeldts Vej 7 - DK-2630 Taastrup - Danemarca
E-mail: info@cortrium.com - Website : www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
46 / 46