Cortrium Apex 1.26

Cortrium Apex 1.26

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
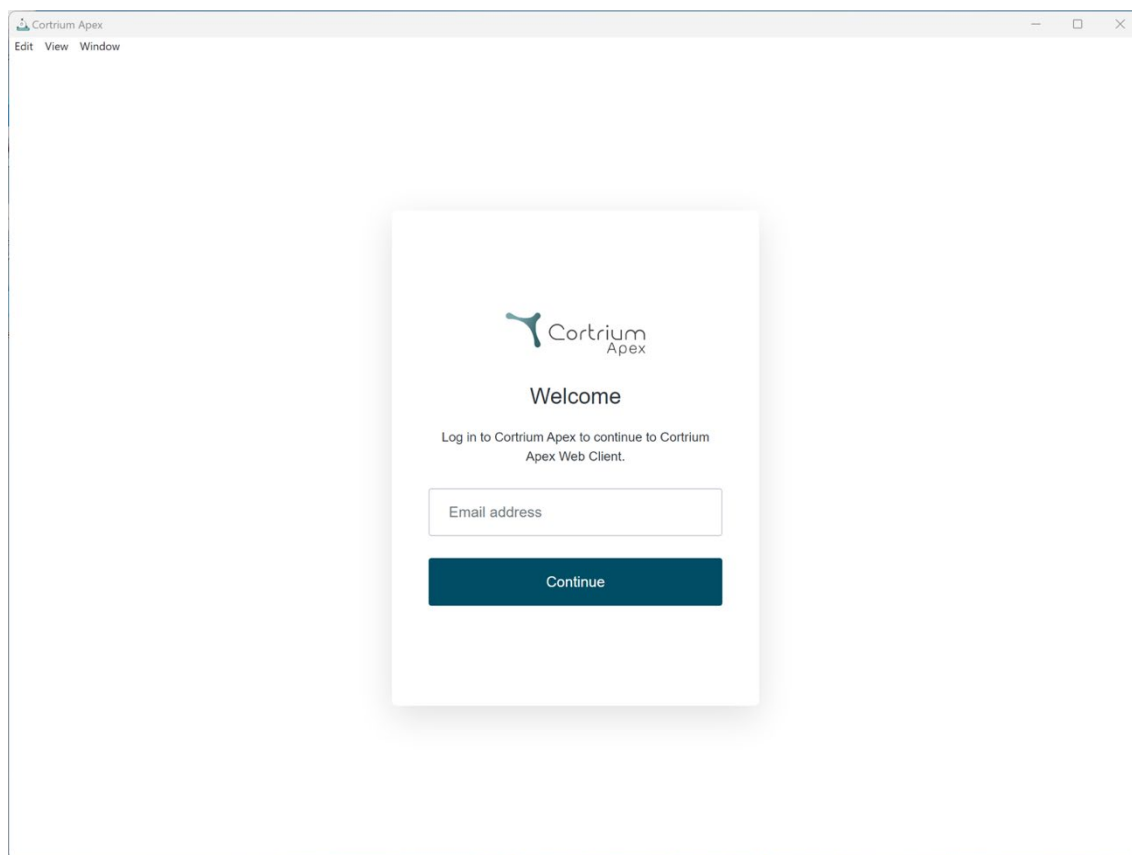
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". In the center of the browser is a white login dialog box. The dialog box features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" below it.

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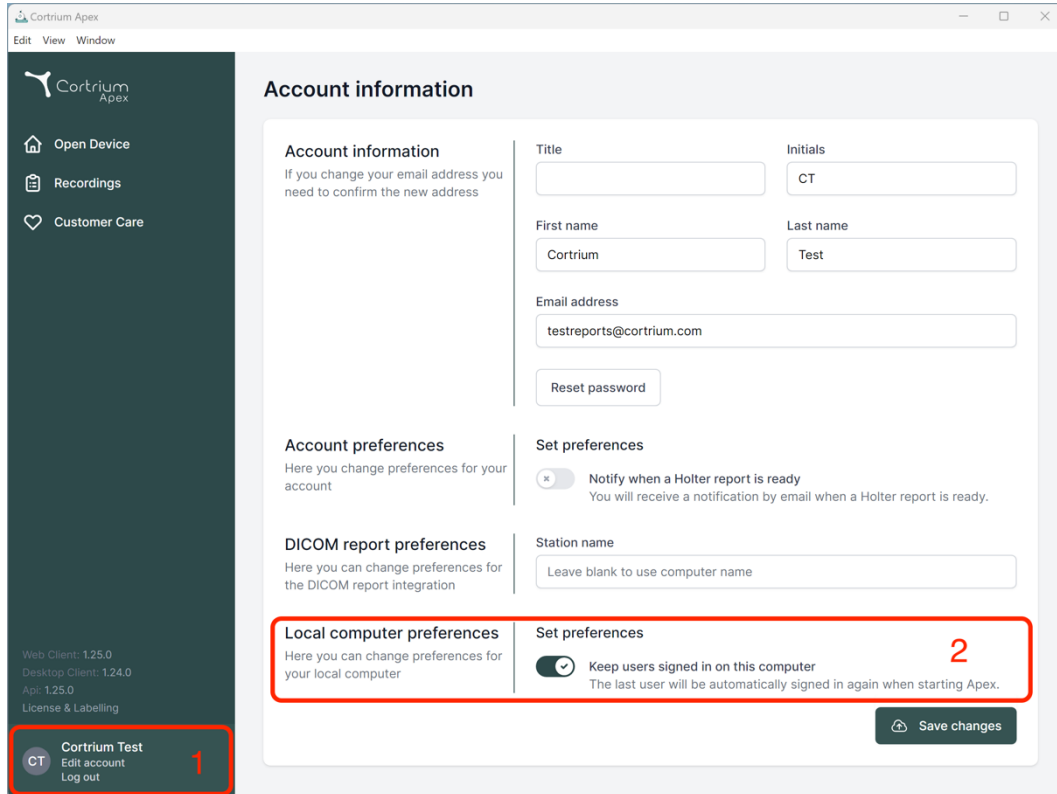The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: Open Device, Recordings, and Customer Care. Below these are version numbers for Web Client, Desktop Client, and API, along with a License & Labelling link. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to Edit account and Log out. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' with a toggle for notifications; 'DICOM report preferences' with a field for Station name; and 'Local computer preferences' with a toggle for 'Keep users signed in on this computer'. A 'Save changes' button is at the bottom right. Red boxes highlight the 'Local computer preferences' section (labeled with a red '1') and the 'Set preferences' section (labeled with a red '2').

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

Local computer preferences

Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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
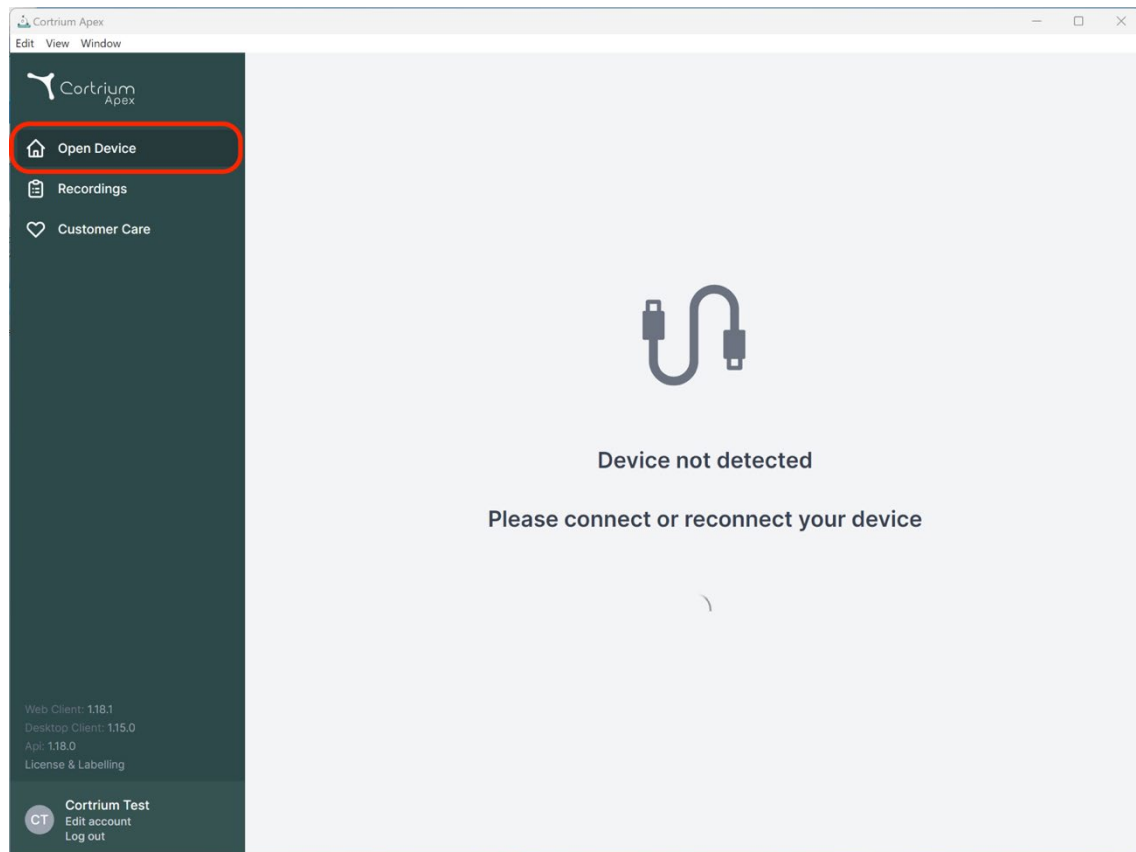
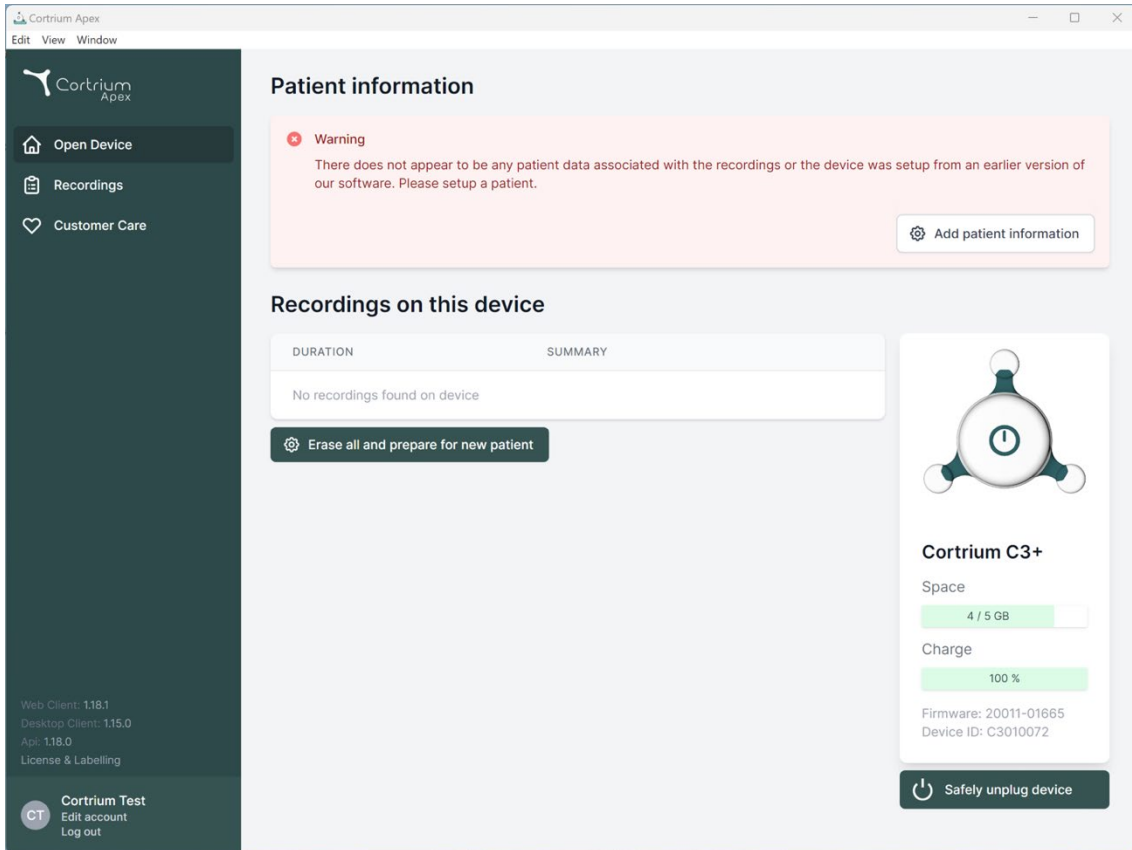
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a user profile section for 'Cortrium Test' with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'DURATION' and 'SUMMARY' columns, currently displaying 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB and 'Charge' at 100 %, along with 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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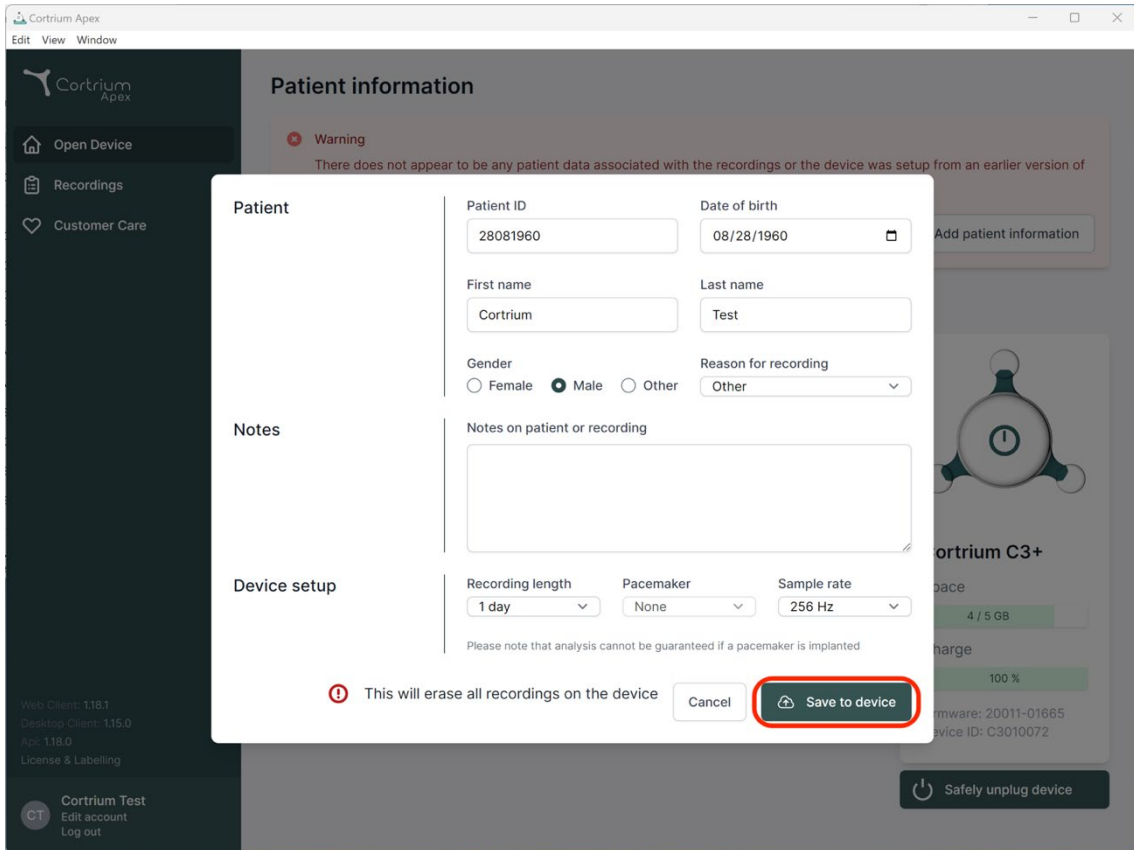Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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
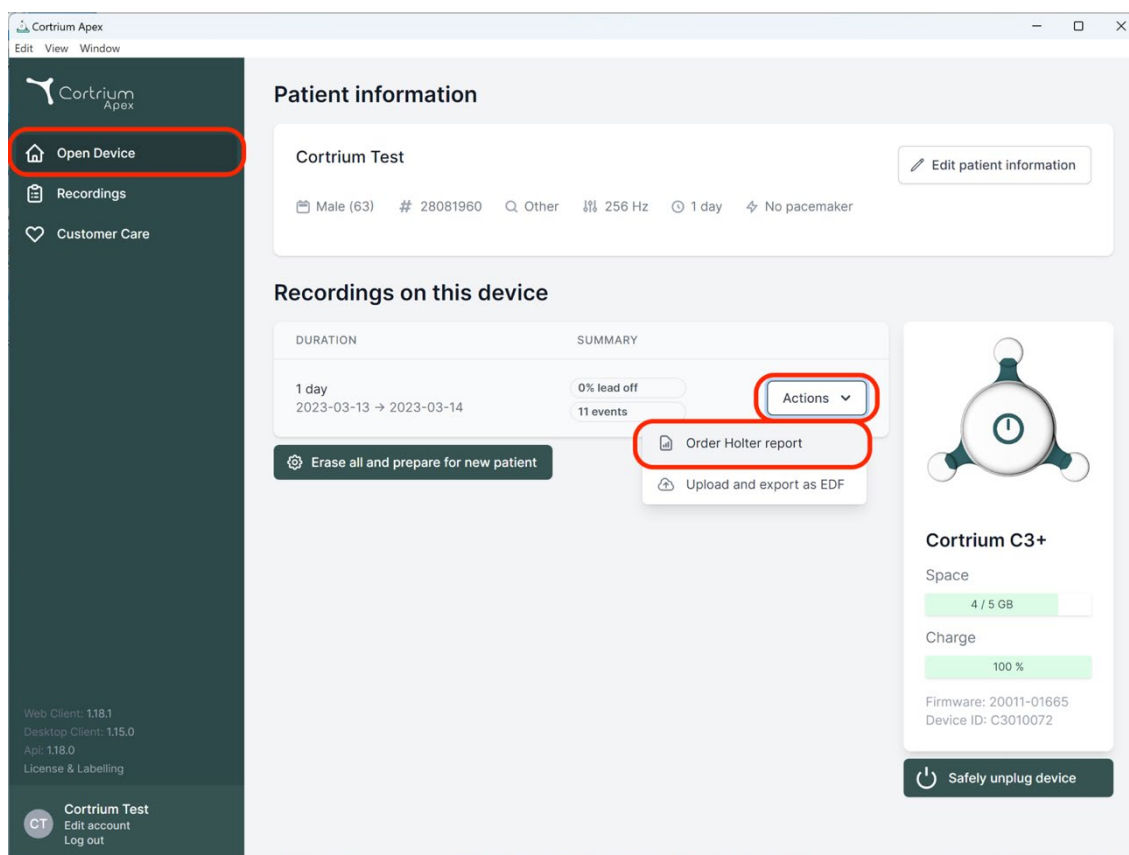
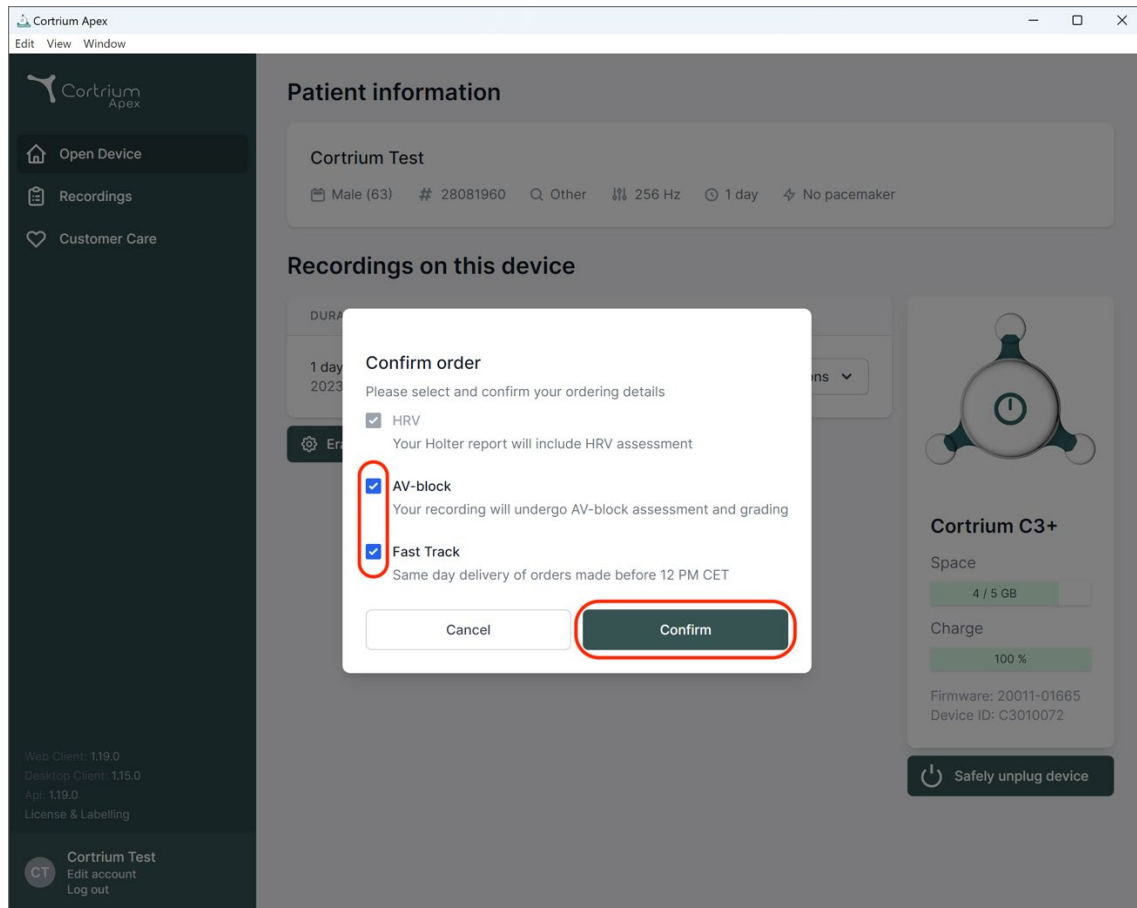
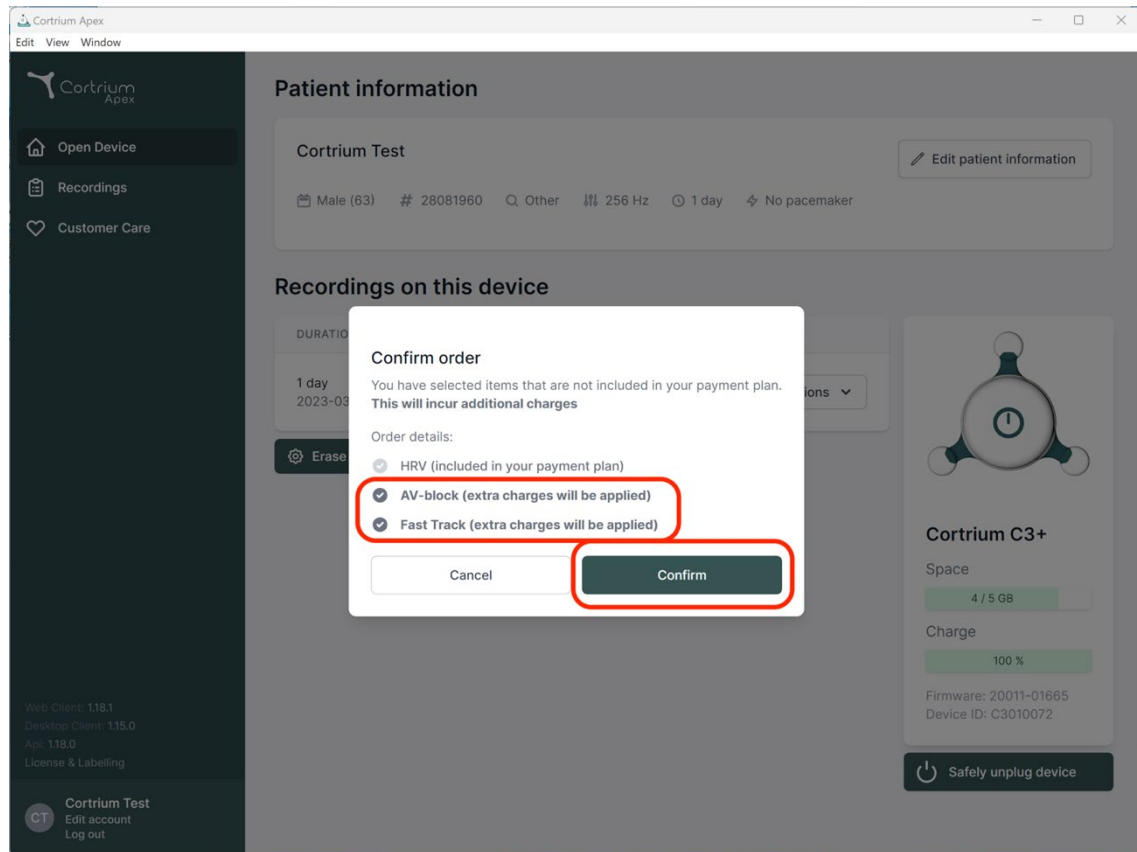
Inspelningen laddas nu upp.

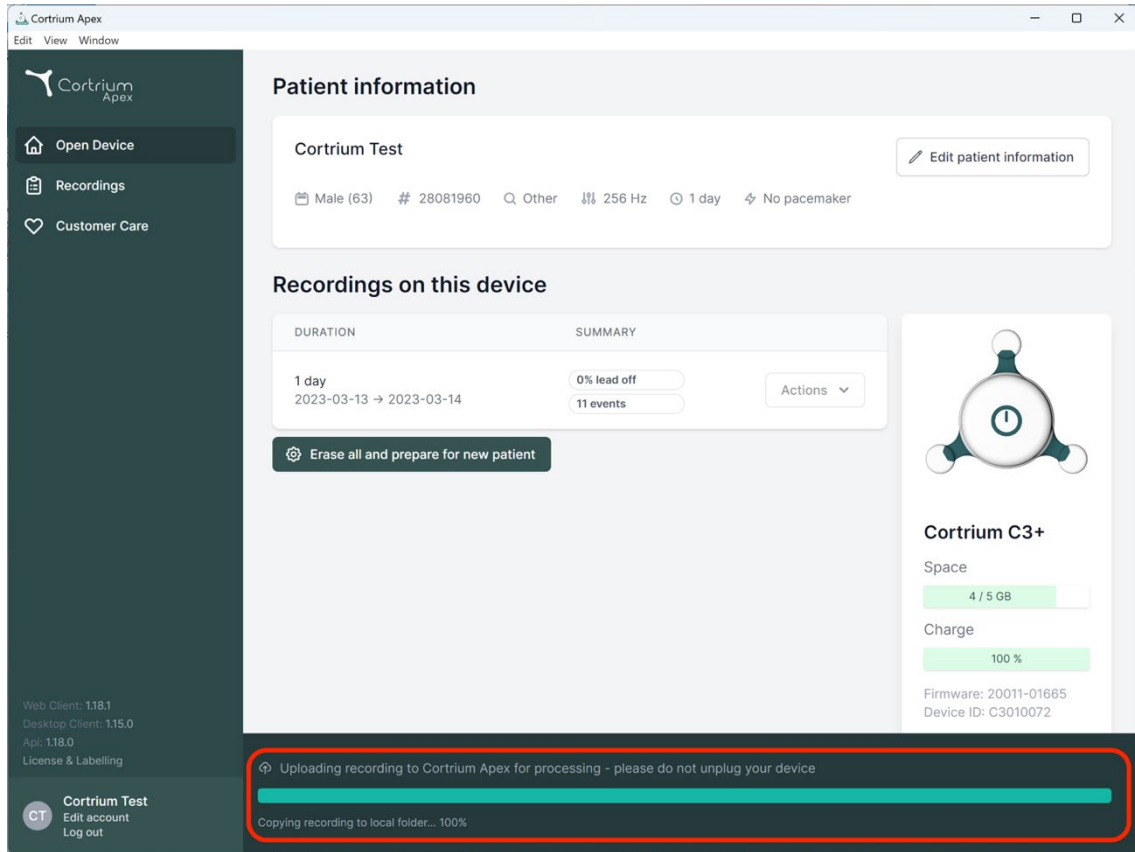The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and recording duration (1 day). Below this is a section for 'Recordings on this device' with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. On the right, there's a device status card for 'Cortrium C3+' showing 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' with a progress bar and 'Copying recording to local folder... 100%'.

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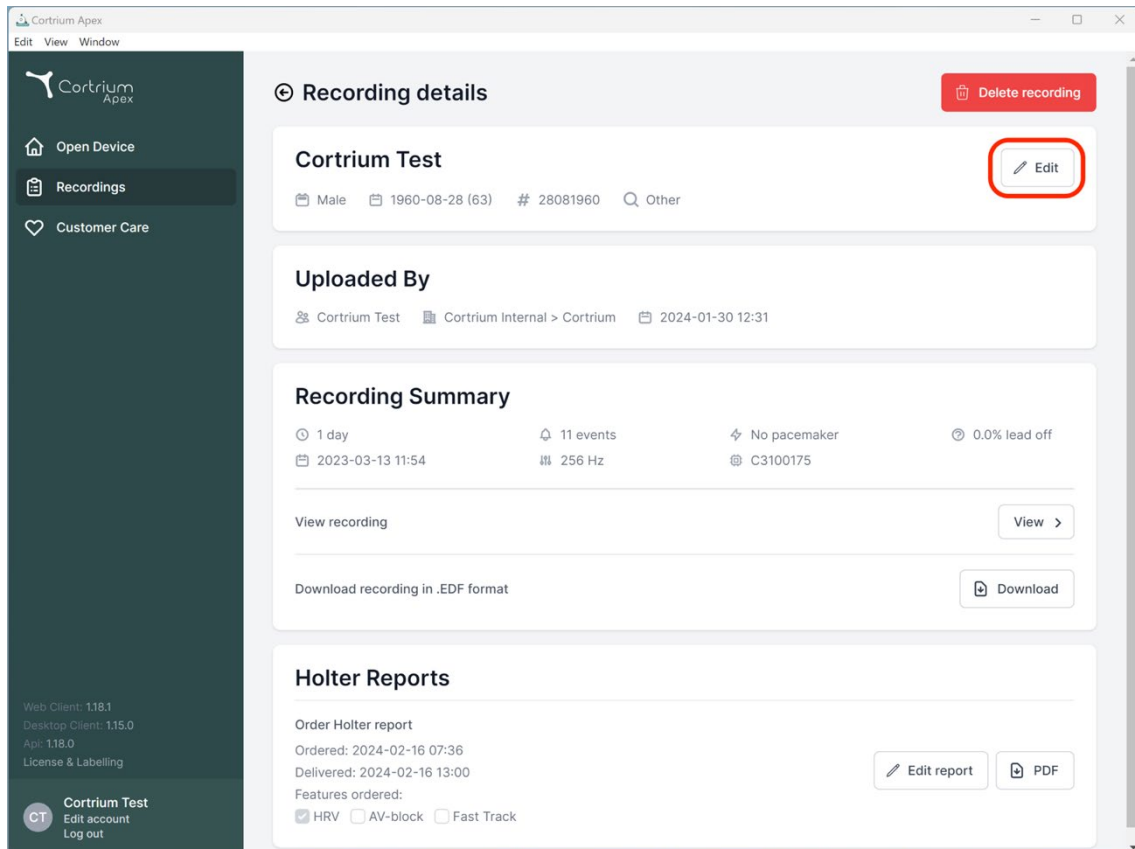
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
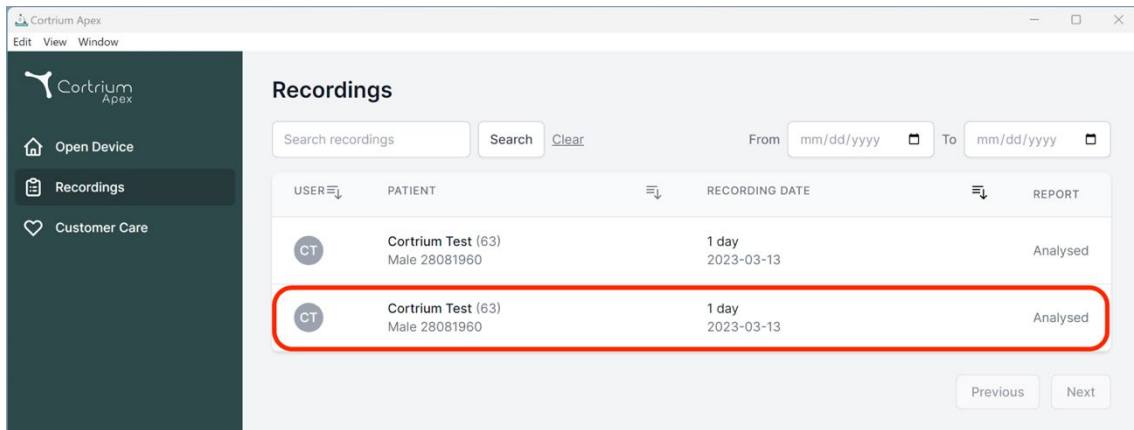
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

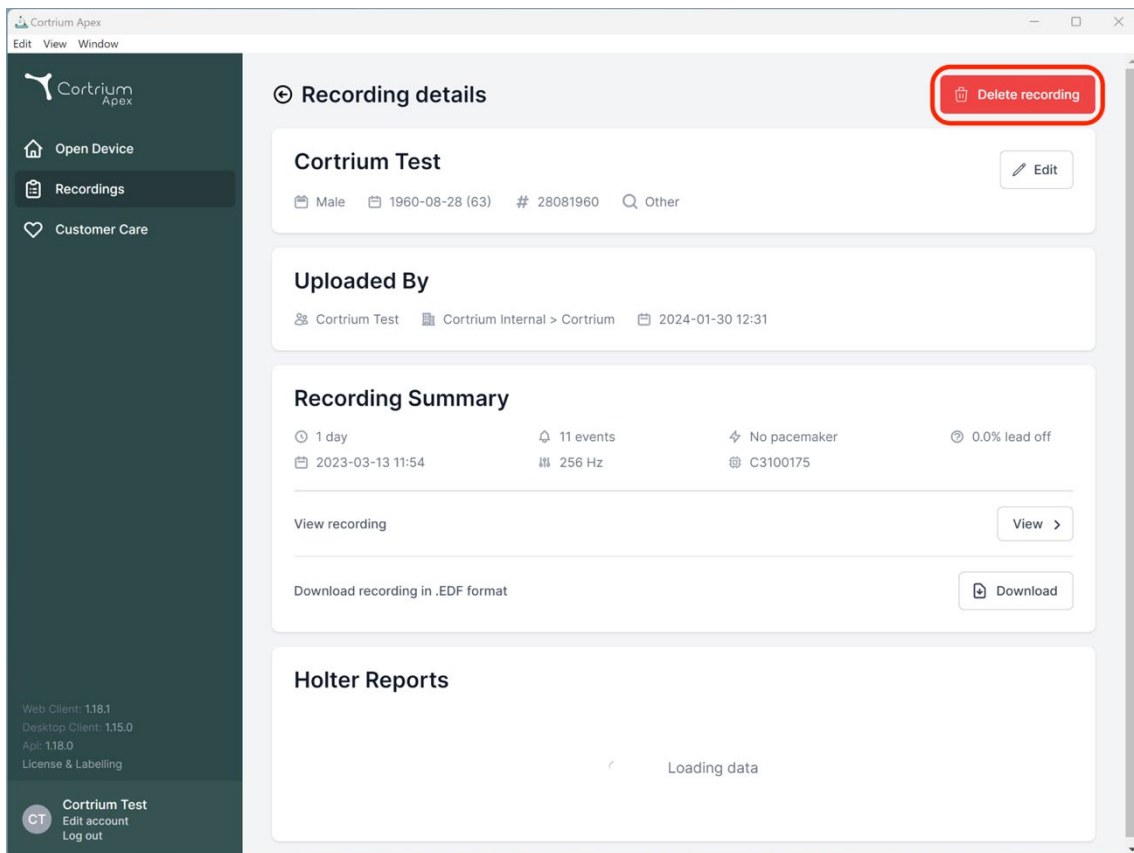Previous Next

Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View](#)

Download recording in .EDF format [Download](#)

Holter Reports

Loading data

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

Cortrium Test 

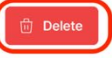
 Male  1960-08-28 (63)  # 28081960  Other


Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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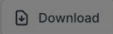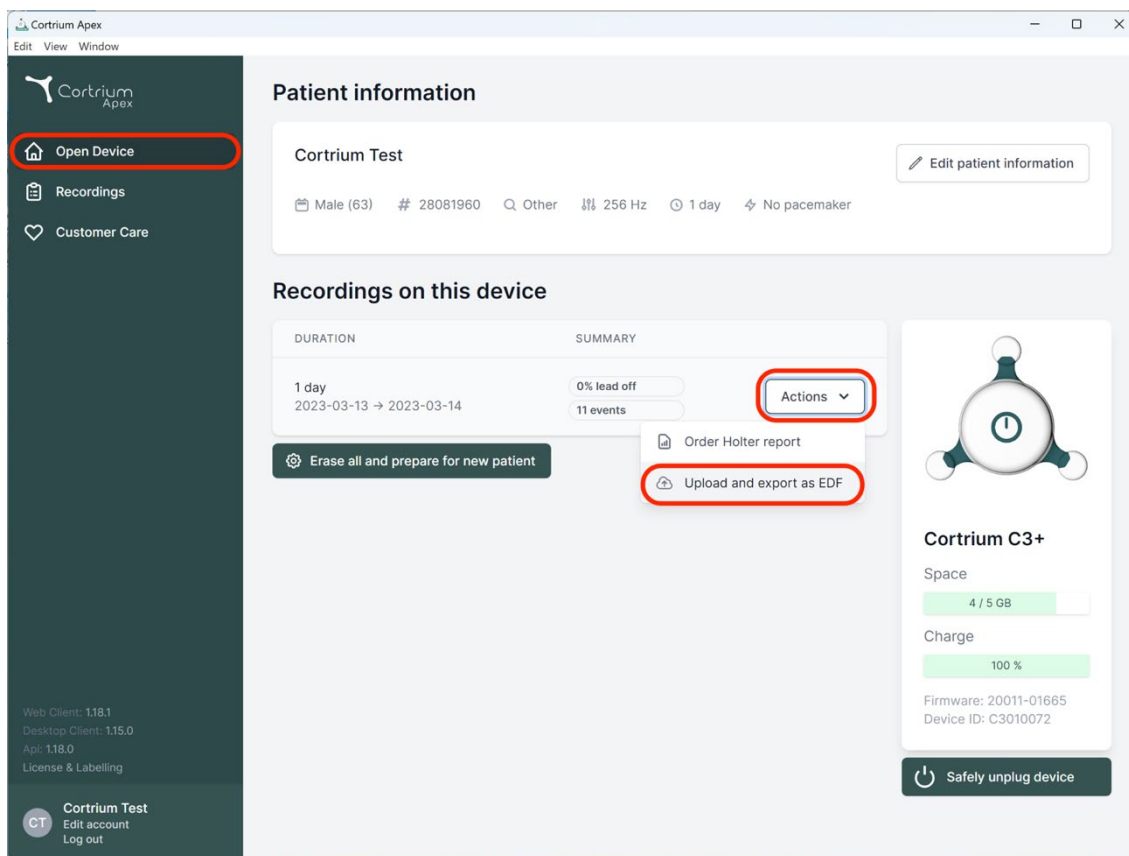
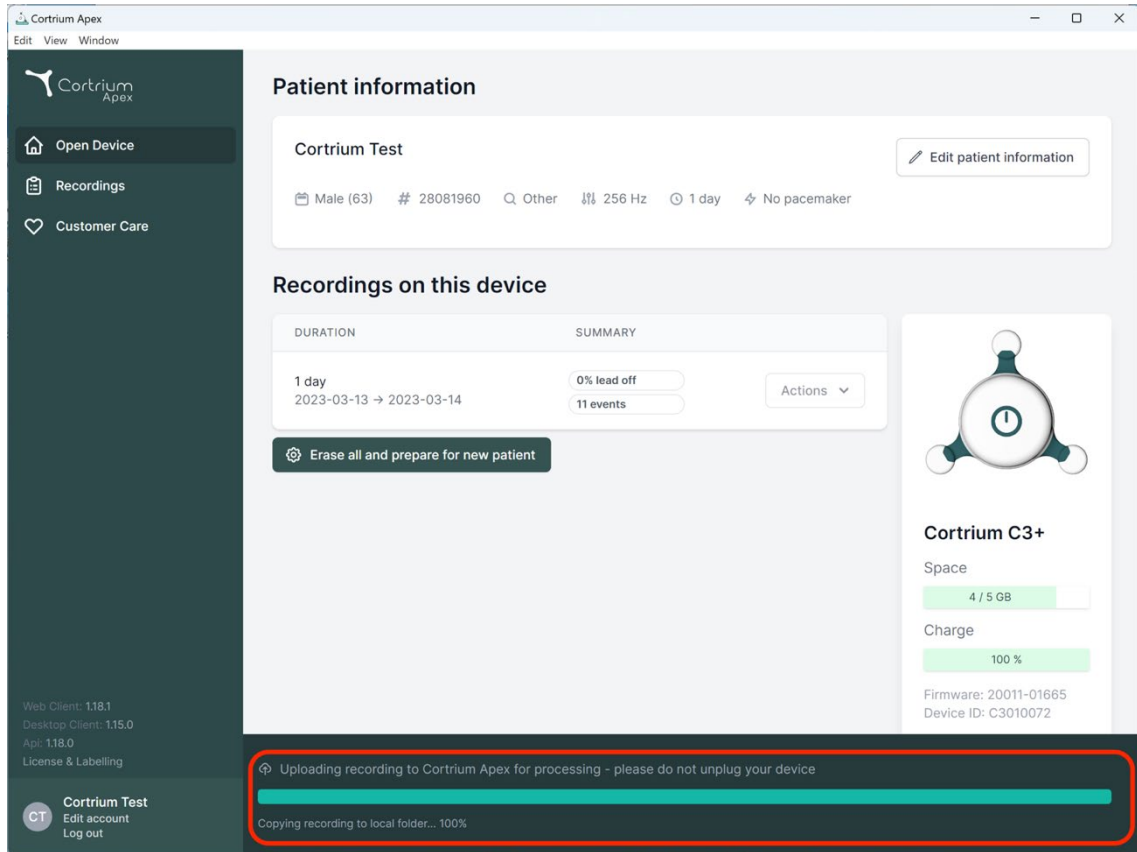
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

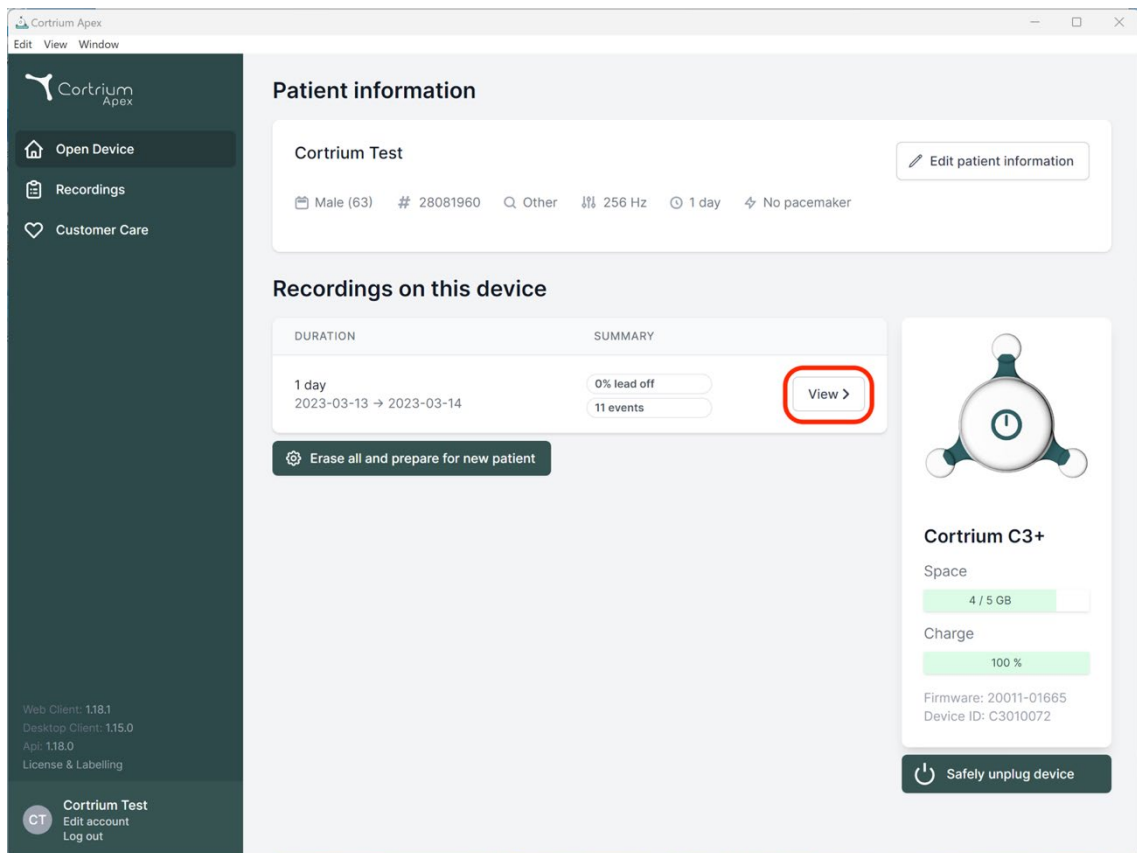
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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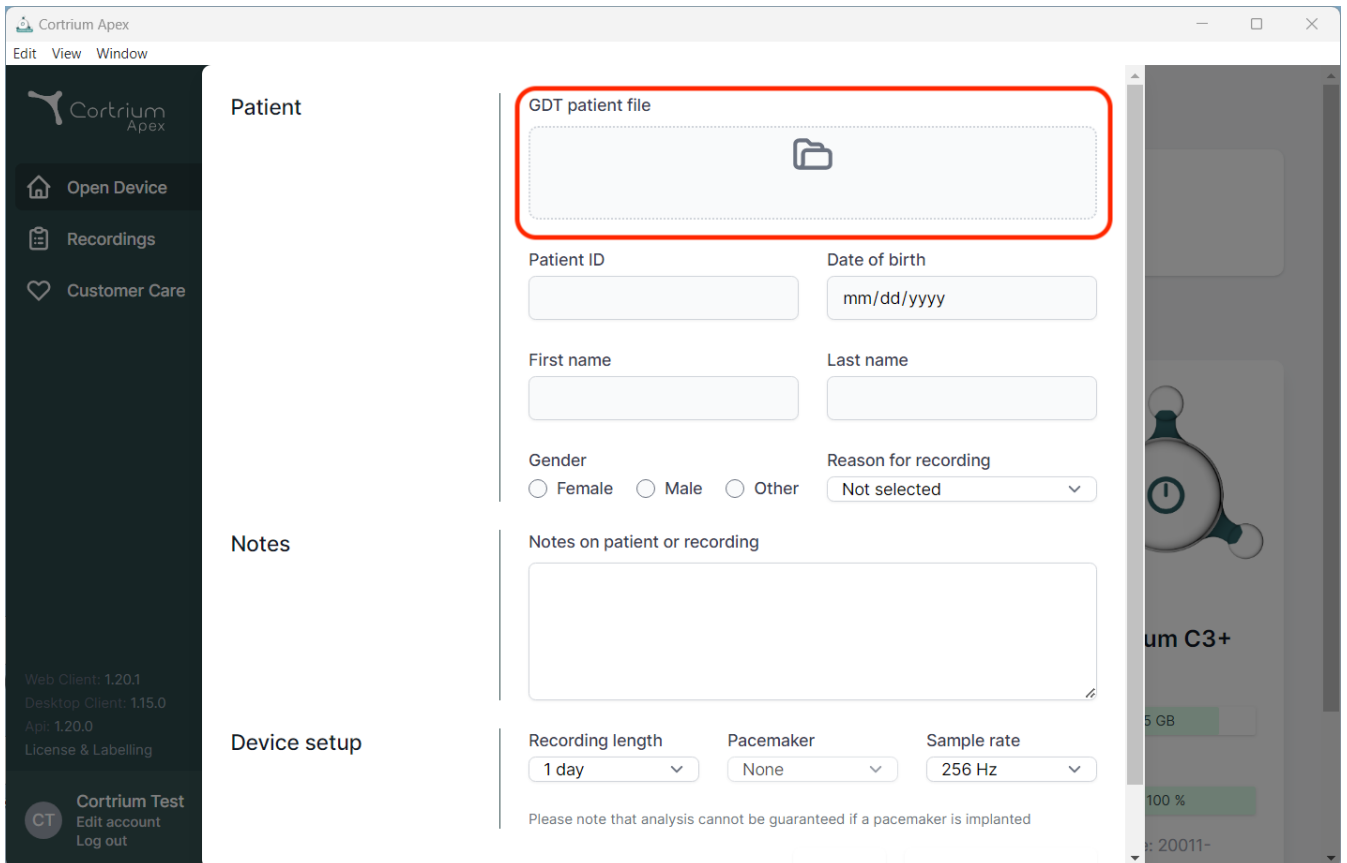
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains the Cortrium Apex logo, navigation links for 'Open Device', 'Recordings', and 'Customer Care', and version information: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', and 'Api: 1.20.0'. The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload field (highlighted with a red box), 'Patient ID' and 'Date of birth' input fields, 'First name' and 'Last name' input fields, 'Gender' radio buttons (Female, Male, Other), and a 'Reason for recording' dropdown menu. The 'Notes' section has a text area for 'Notes on patient or recording'. The 'Device setup' section includes 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz) dropdown menus. A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The right sidebar shows a partial view of a device recording, including a 'um C3+' label and a '5 GB' storage indicator.

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
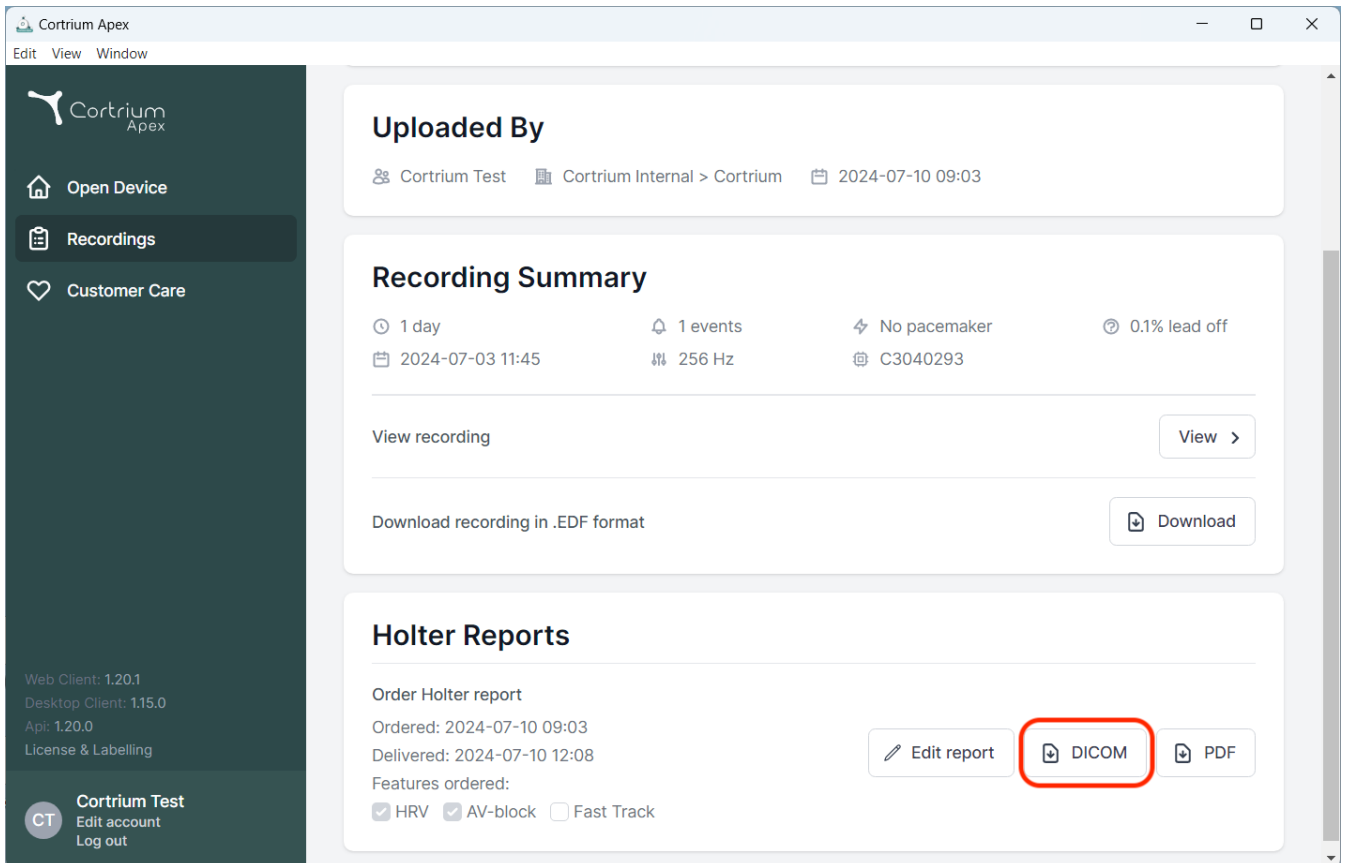
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections. The top section, 'Uploaded By', shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'. The middle section, 'Recording Summary', lists details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'. The bottom section, 'Holter Reports', shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered' as 'HRV', 'AV-block', and 'Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'. The bottom left of the sidebar shows 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', 'License & Labelling', and a user profile for 'Cortrium Test'.

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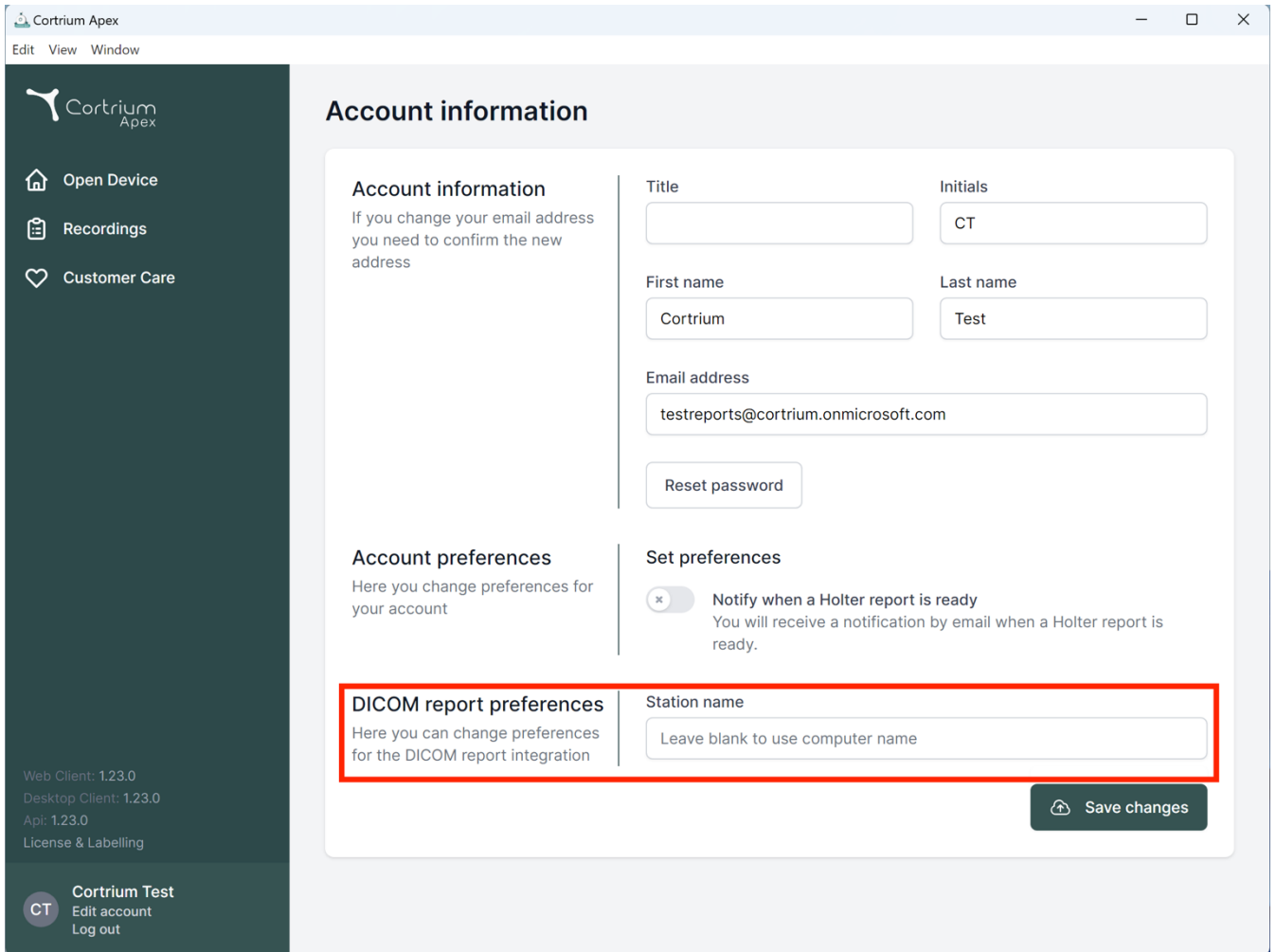Cortrium Apex
Edit View Window

Account information
If you change your email address you need to confirm the new address

Account preferences
Here you change preferences for your account

Set preferences
☐ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name
Leave blank to use computer name

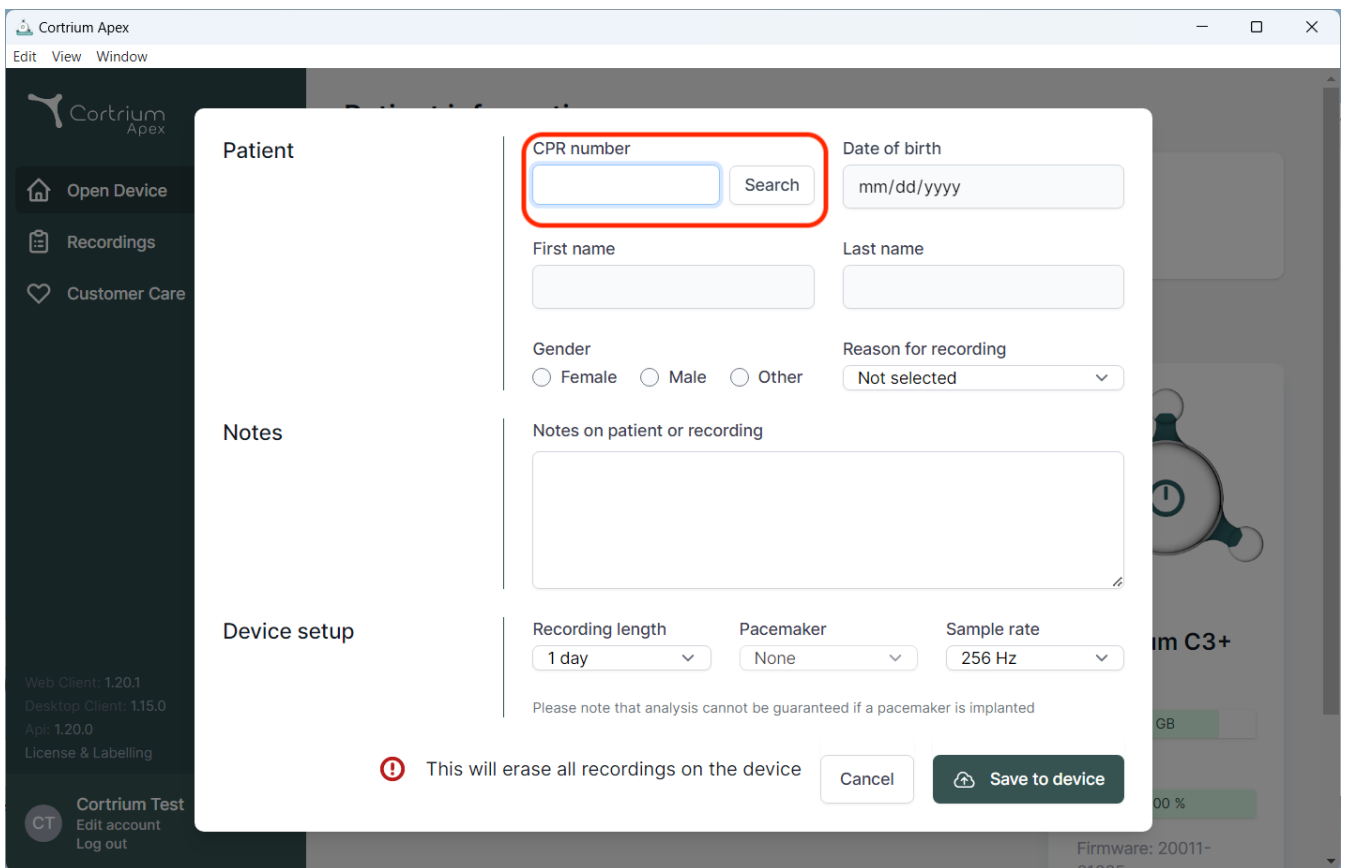
Save changes

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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Pacemaker

Sample rate

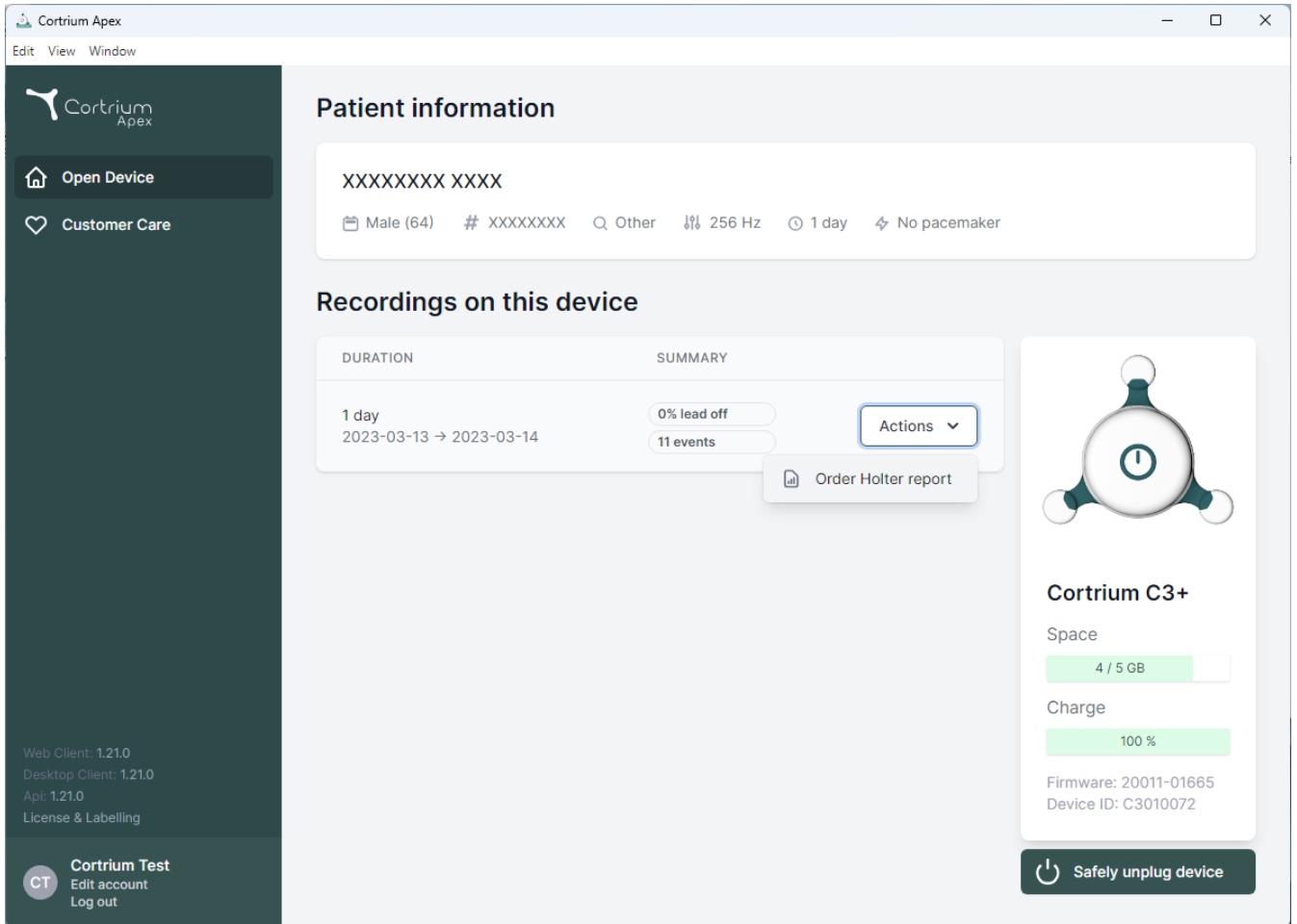
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' options.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section displays a table with two columns: 'DURATION' and 'SUMMARY'. The table shows a recording for '1 day' from '2023-03-13' to '2023-03-14', with '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right side, a device status card for 'Cortrium C3+' shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of the card.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

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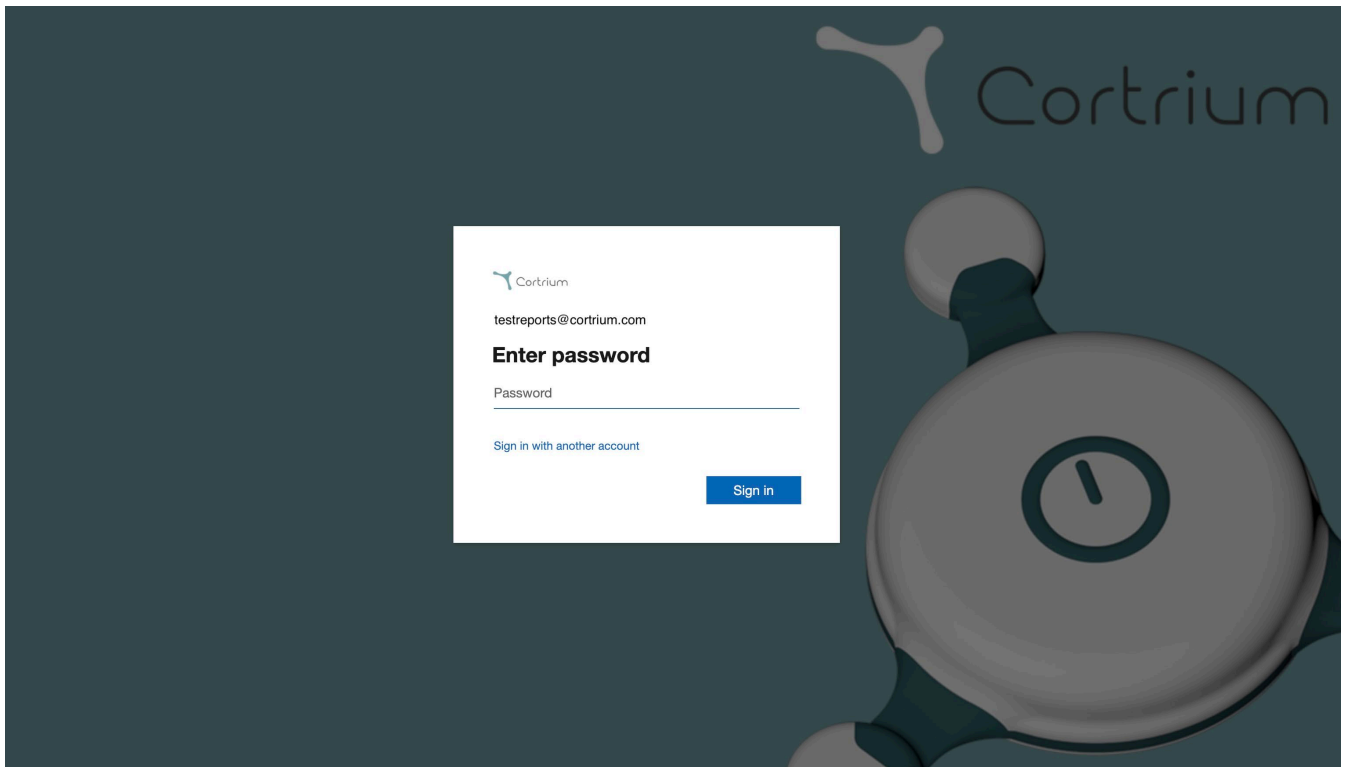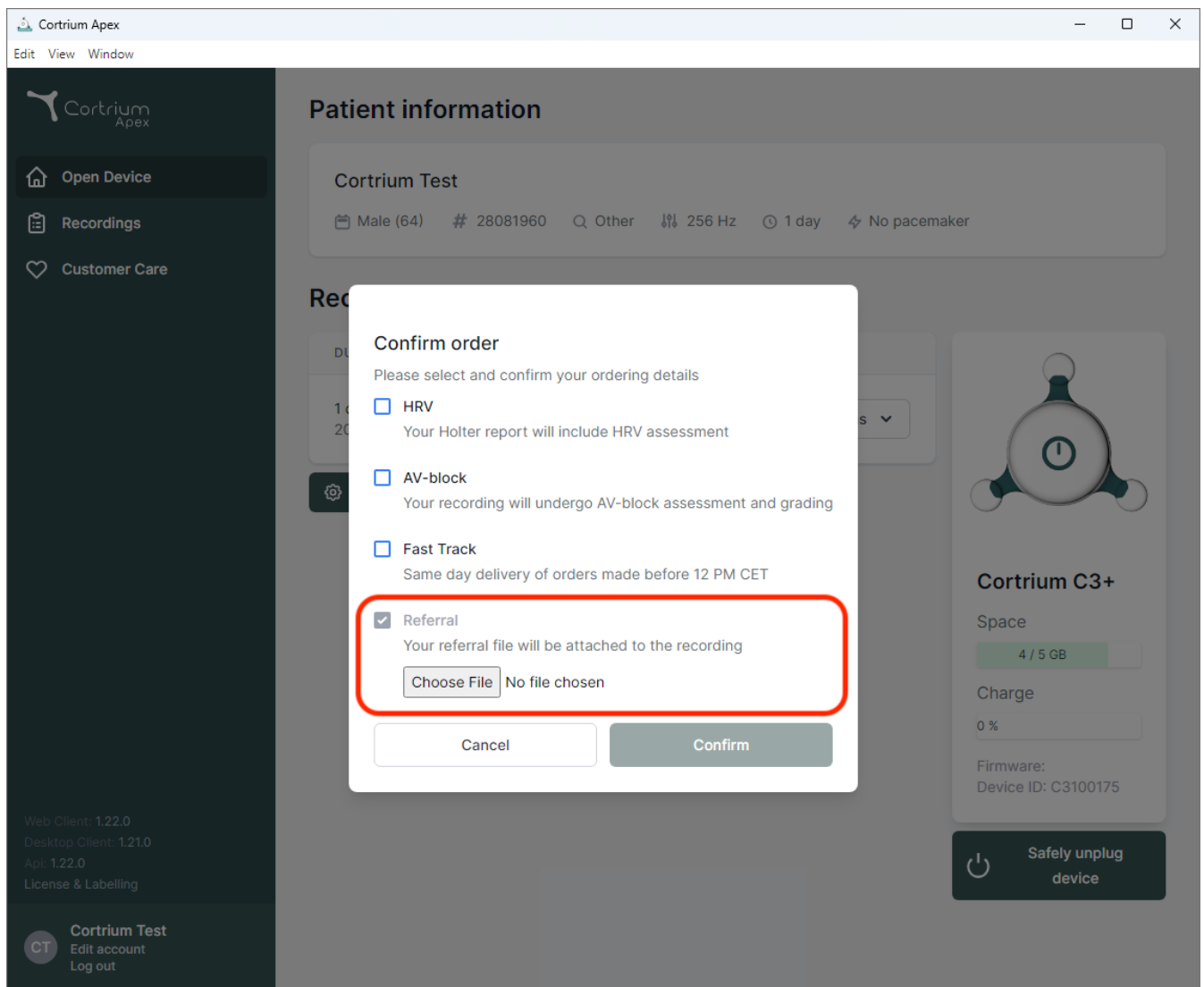
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
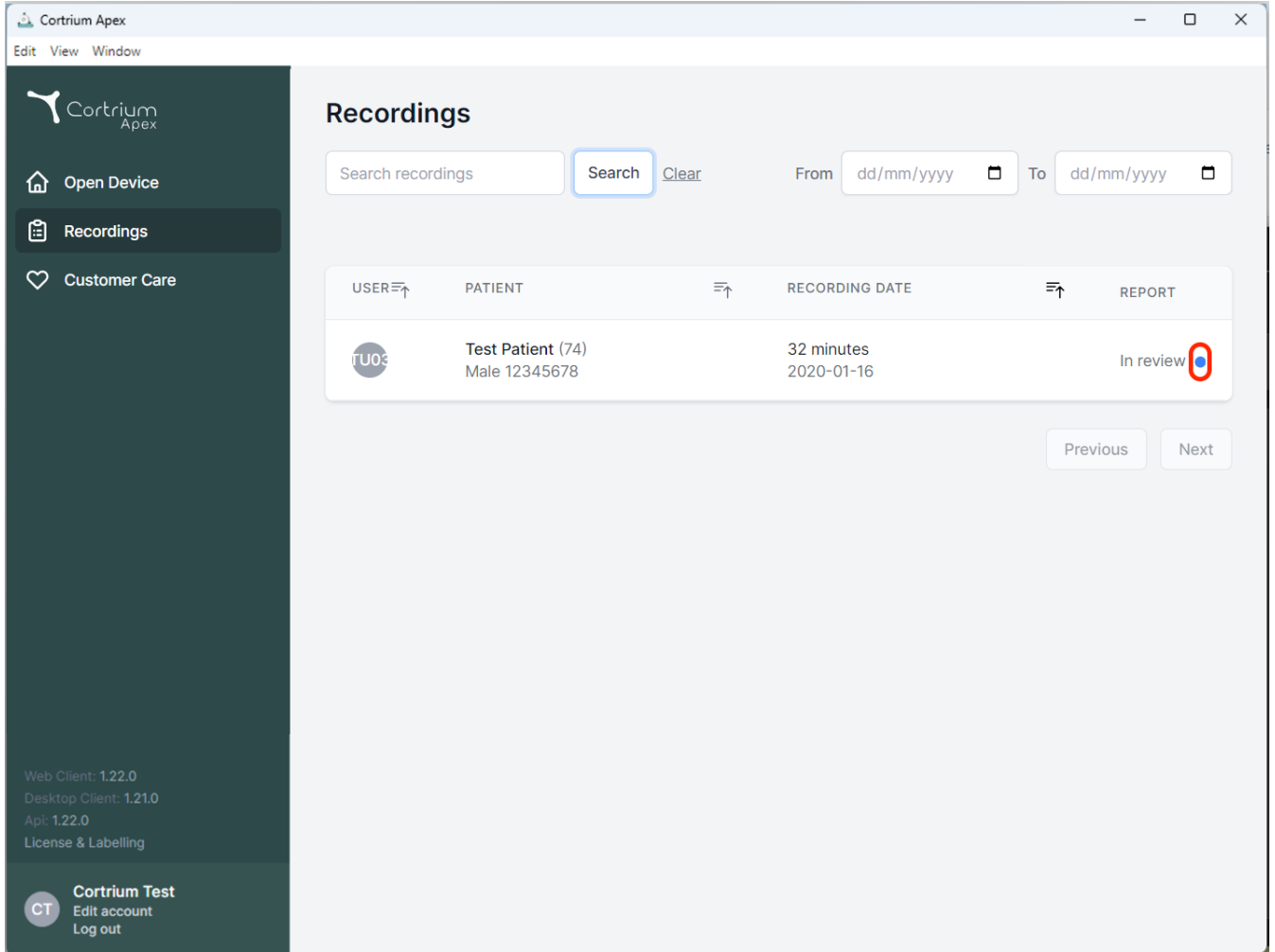
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (selected), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot is visible next to the "In review" status, indicating a pending report. The bottom of the sidebar shows version information and a user profile for "Cortrium Test".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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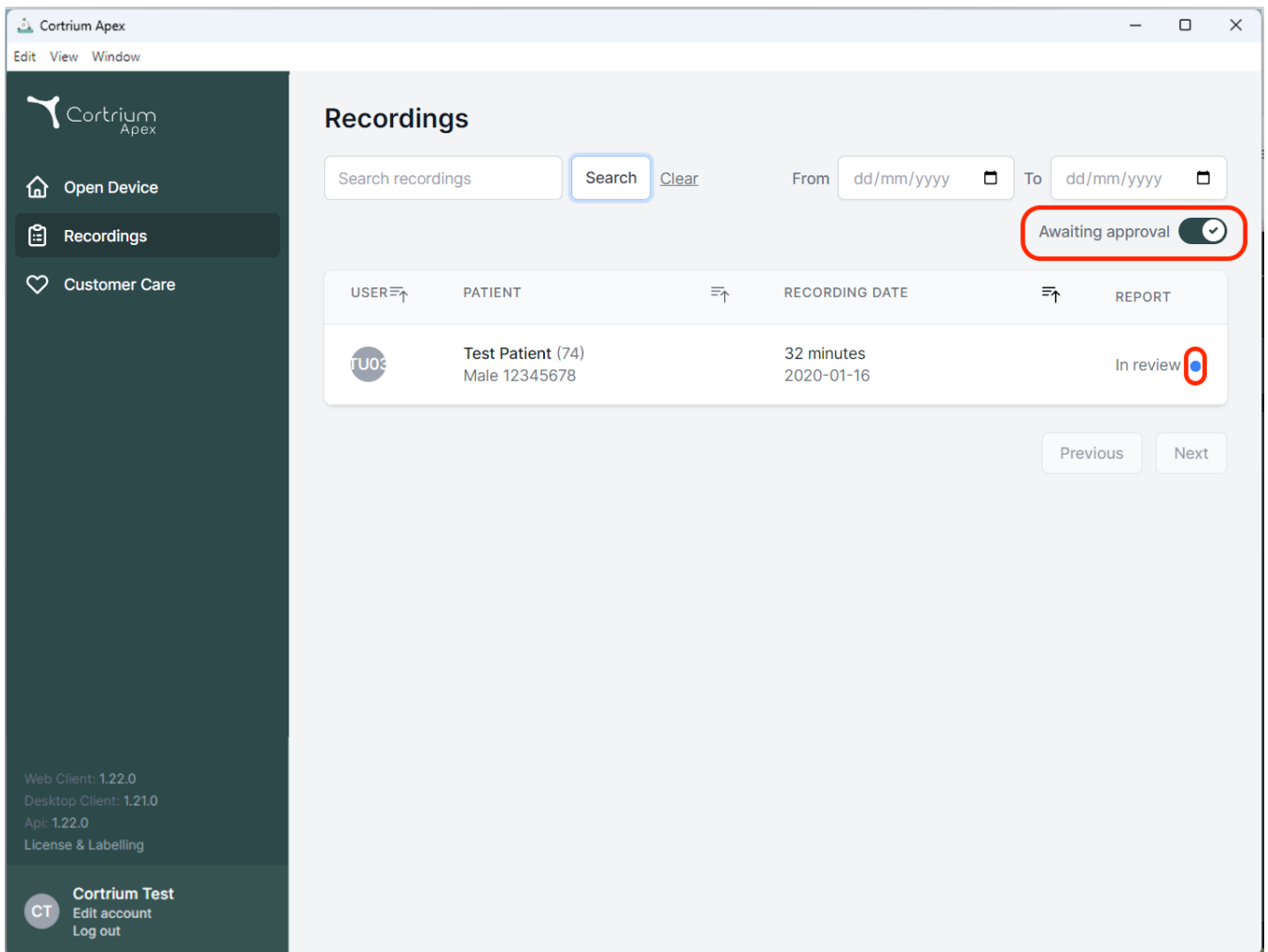Edit View Window

Recordings

Search recordings Search Clear From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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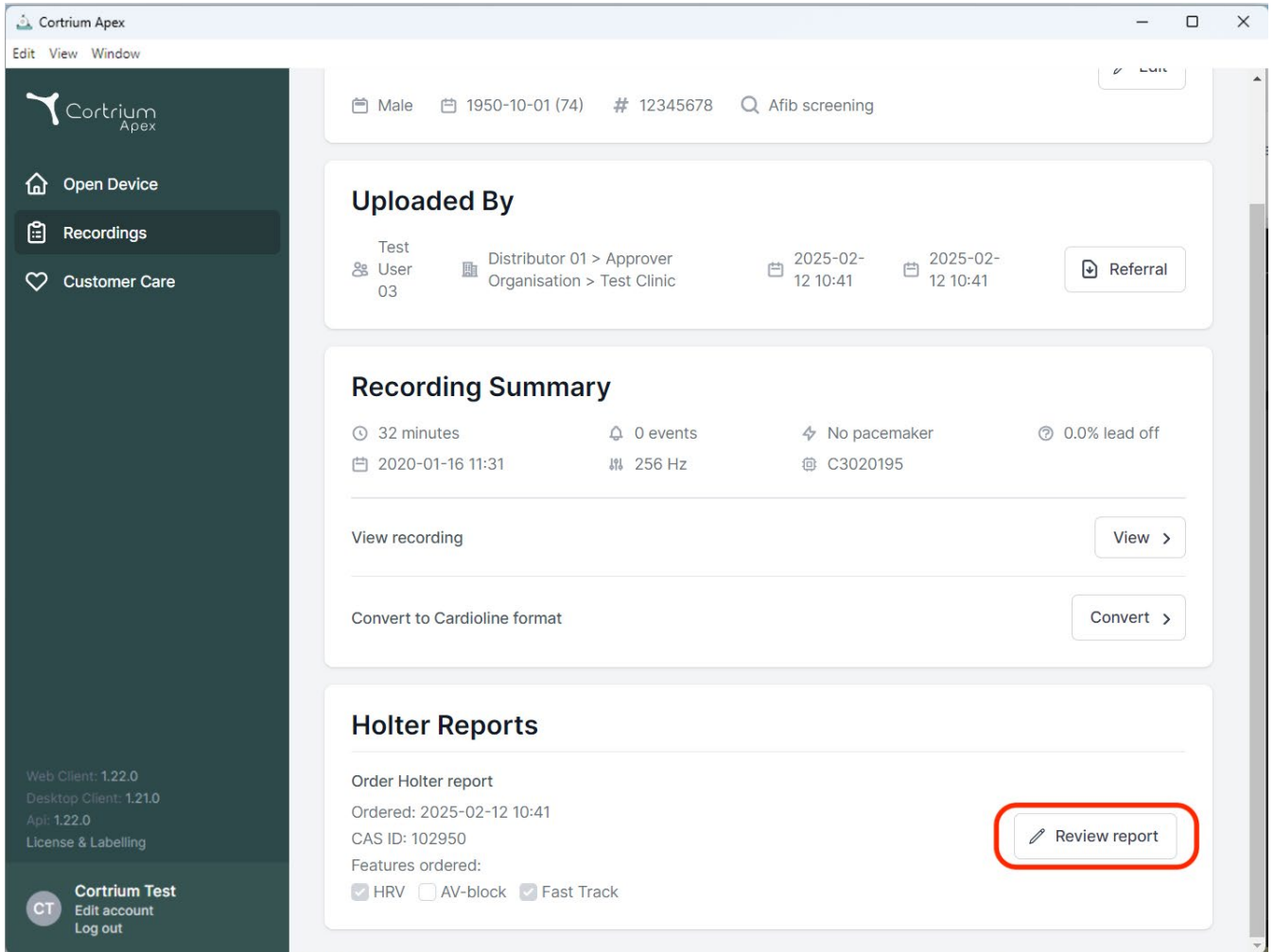
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and three main menu items: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and upload dates (2025-02-12 10:41). A 'Referral' button is present.
- Recording Summary:** A section providing key recording metrics: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date (2020-01-16 11:31) and frequency (256 Hz). Buttons for 'View recording' and 'Convert to Cardioline format' are available.
- Holter Reports:** A section for managing holter reports. It shows the 'Order Holter report' status, ordered date (2025-02-12 10:41), and CAS ID (102950). Under 'Features ordered', 'HRV' and 'Fast Track' are selected, while 'AV-block' is not. A 'Review report' button is highlighted with a red rectangle.

At the bottom left, version information is displayed: Web Client: 1.22.0, Desktop Client: 1.21.0, and Apl: 1.22.0. The bottom right corner shows the user 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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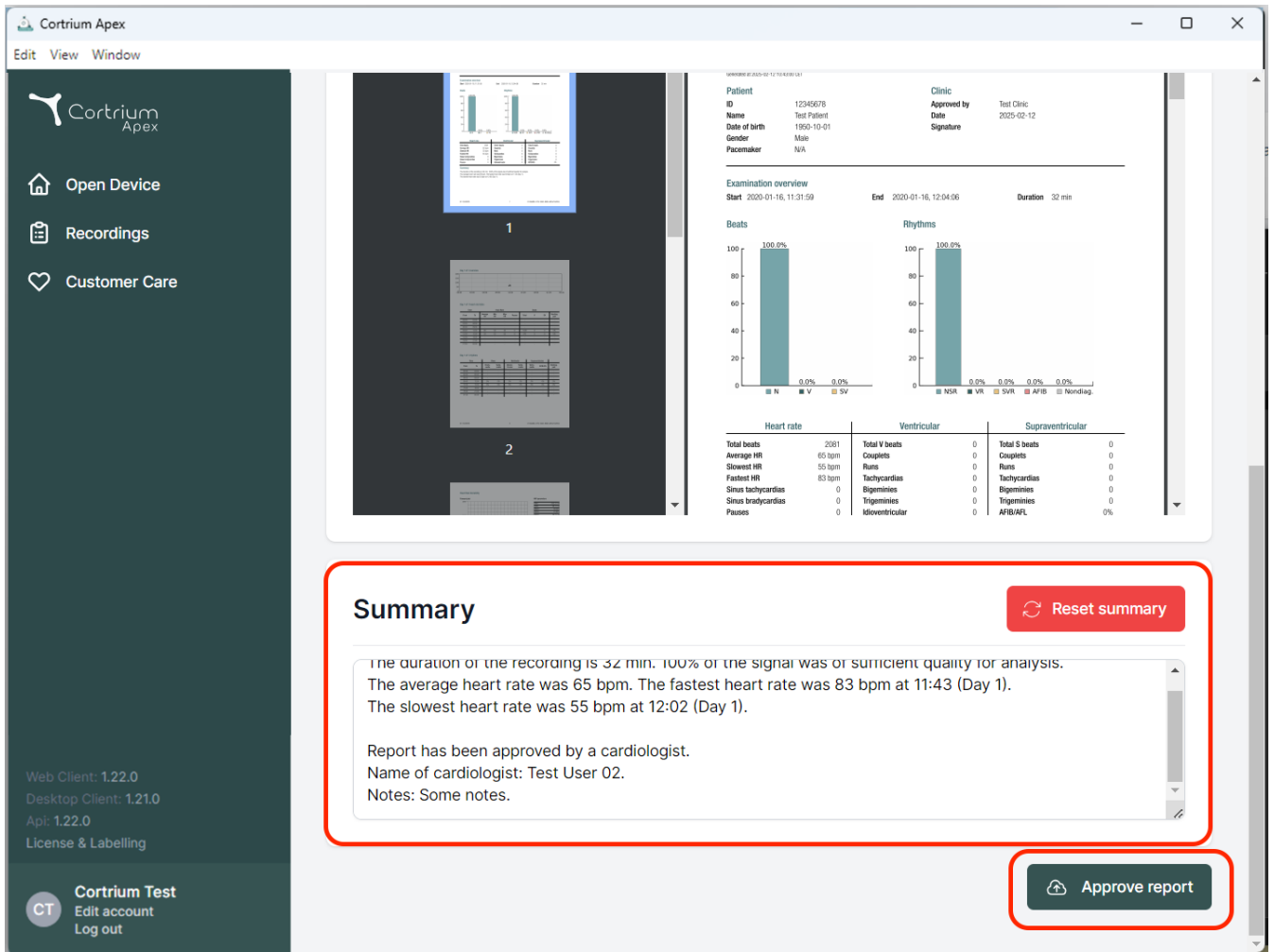Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient
ID: 12345678
Name: Test Patient
Date of birth: 1950-10-01
Gender: Male
Pacemaker: N/A

Clinic
Approved by: Test Clinic
Date: 2025-02-12
Signature:

Examination overview
Start: 2020-01-16, 11:31:59
End: 2020-01-16, 12:04:06
Duration: 32 min

Beats
100.0%
0.0%
0.0%
0.0%

Rhythms
100.0%
0.0%
0.0%
0.0%
0.0%
0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AF/IB/ATL	0%

Summary Reset summary

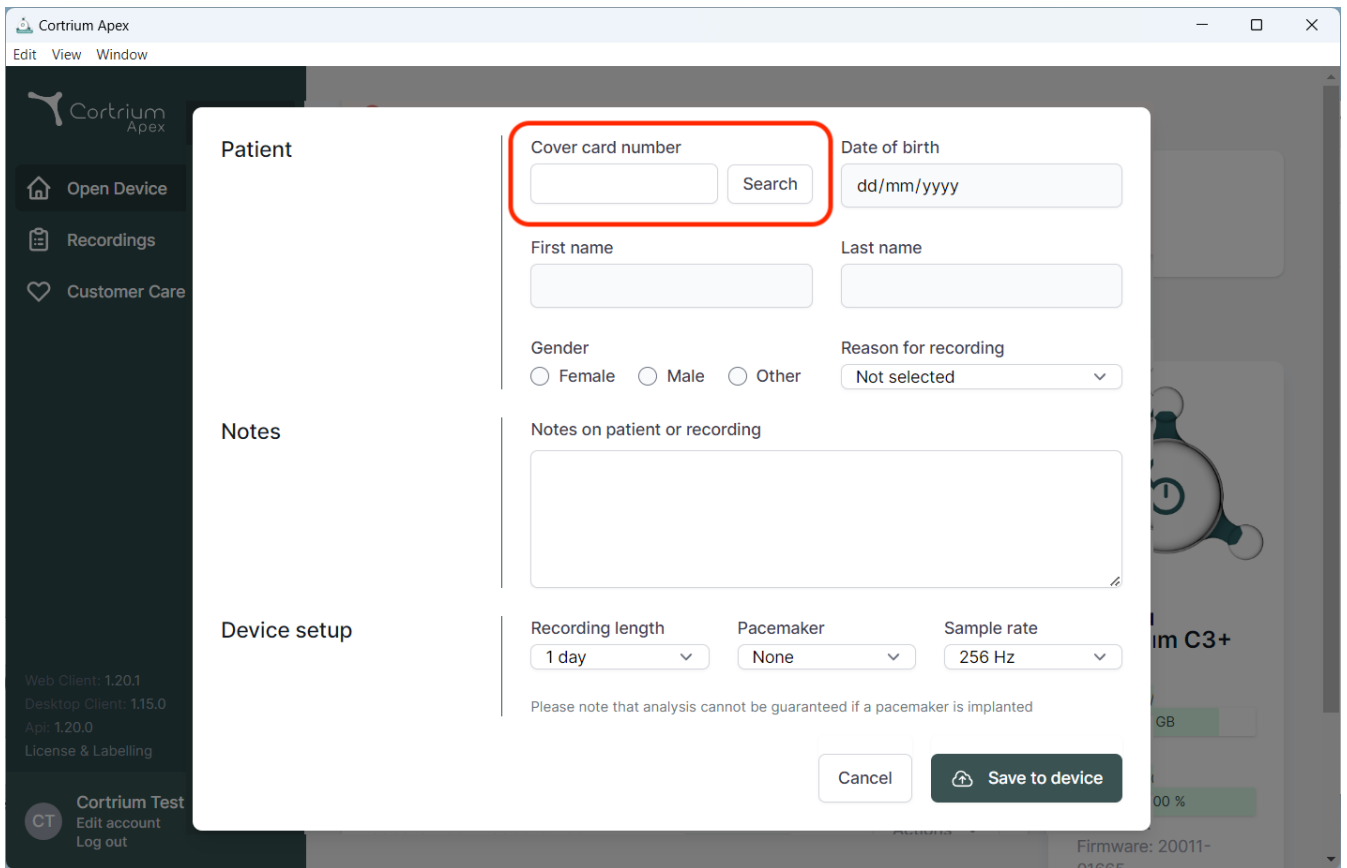
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying various input fields for patient information. The "Cover card number" field is highlighted with a red rectangle. The "Search" button is also visible. Other fields include "Date of birth" (dd/mm/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), and "Notes on patient or recording". The "Device setup" section includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". Buttons for "Cancel" and "Save to device" are at the bottom right.

Patient

Cover card number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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