Cortrium Apex 1.26

Cortrium Apex 1.26

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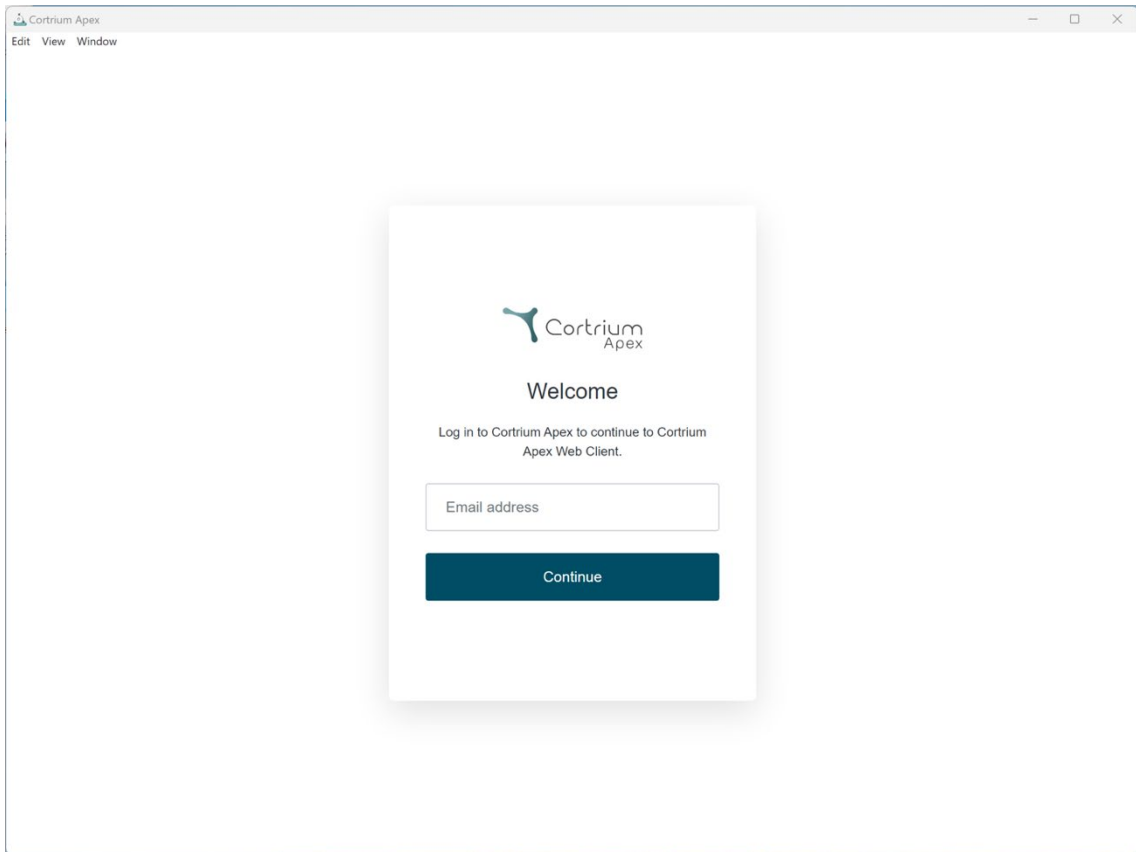
 Cortrium
Apex

Welcome

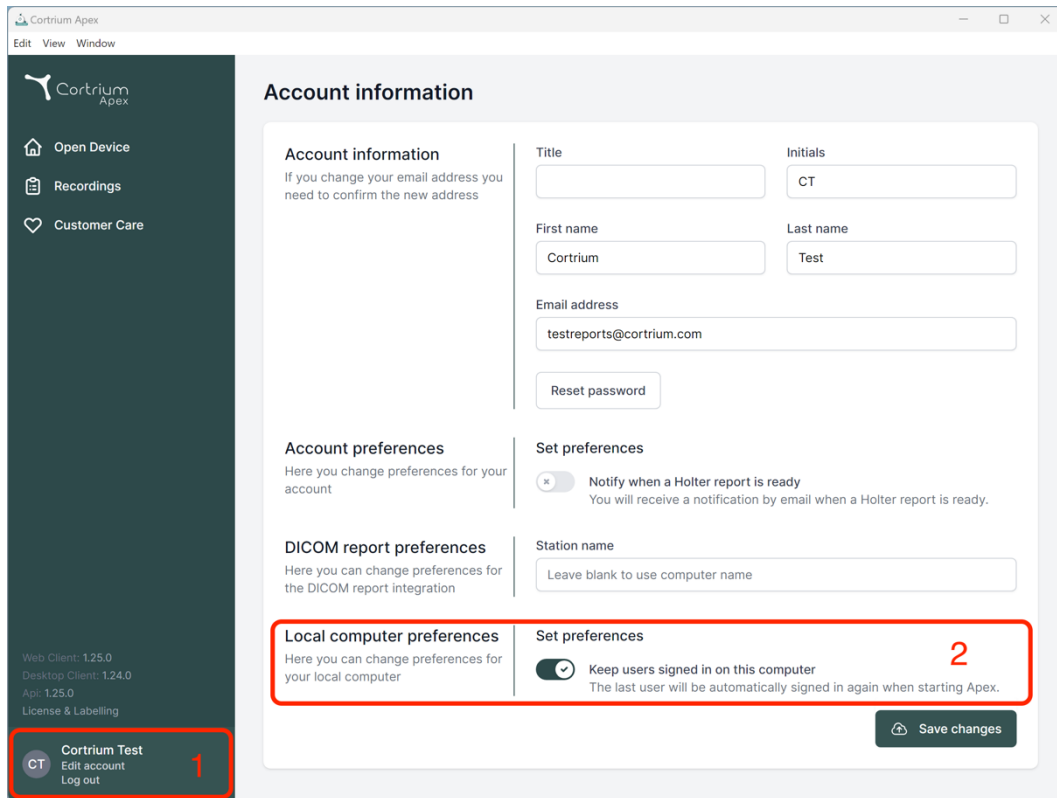
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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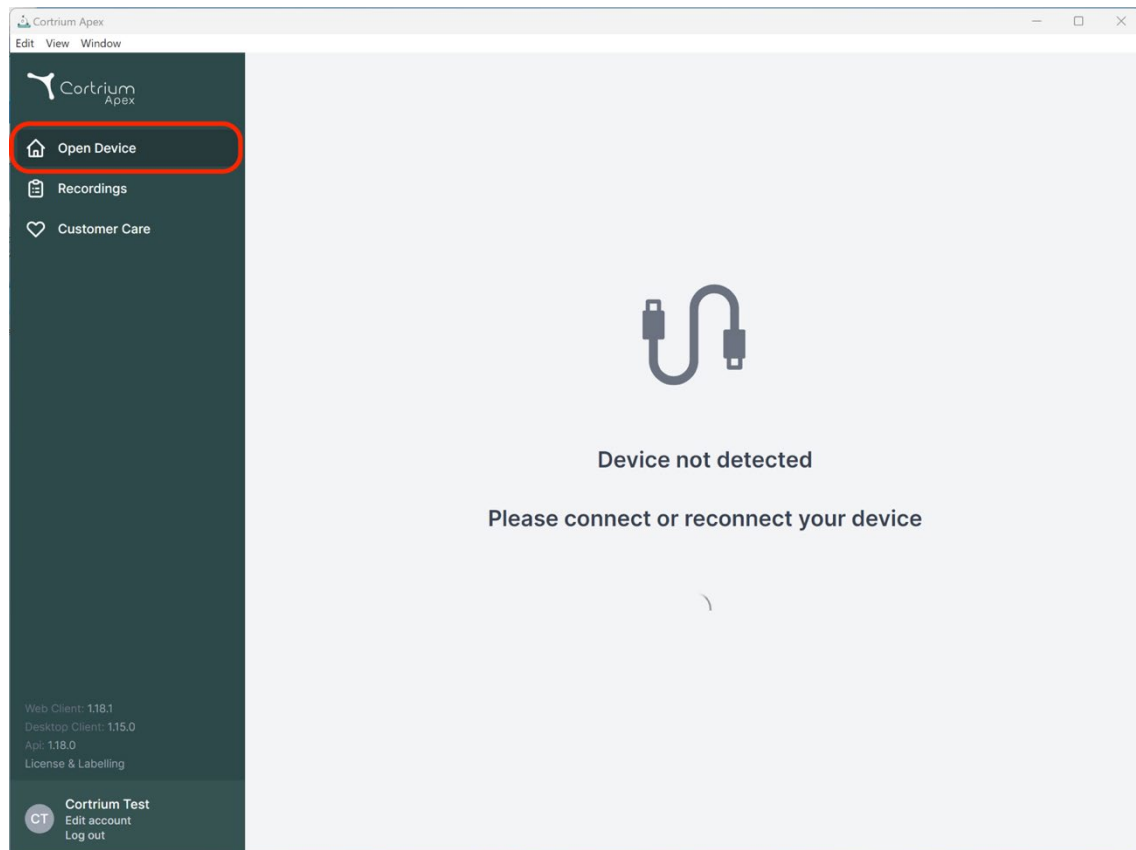
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar is a user profile card for 'Cortrium Test' with an 'Edit account' link and a 'Log out' button. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, First name, Last name, and Email address), 'Account preferences' (with a toggle for 'Notify when a Holter report is ready'), 'DICOM report preferences' (with a field for 'Station name'), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer'). The 'Local computer preferences' section is highlighted with a red box and a red number '1'. The 'Keep users signed in on this computer' toggle is also highlighted with a red box and a red number '2'. A 'Save changes' button is located at the bottom right of the main content area.

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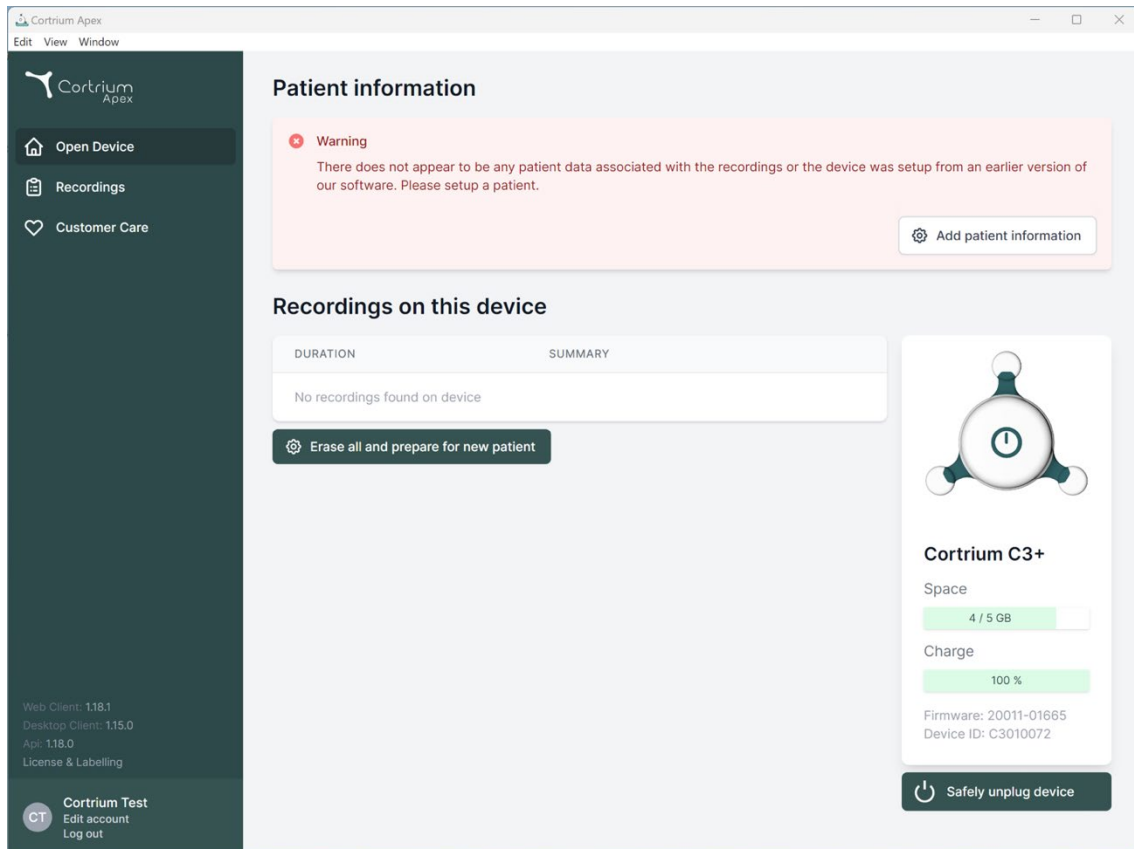
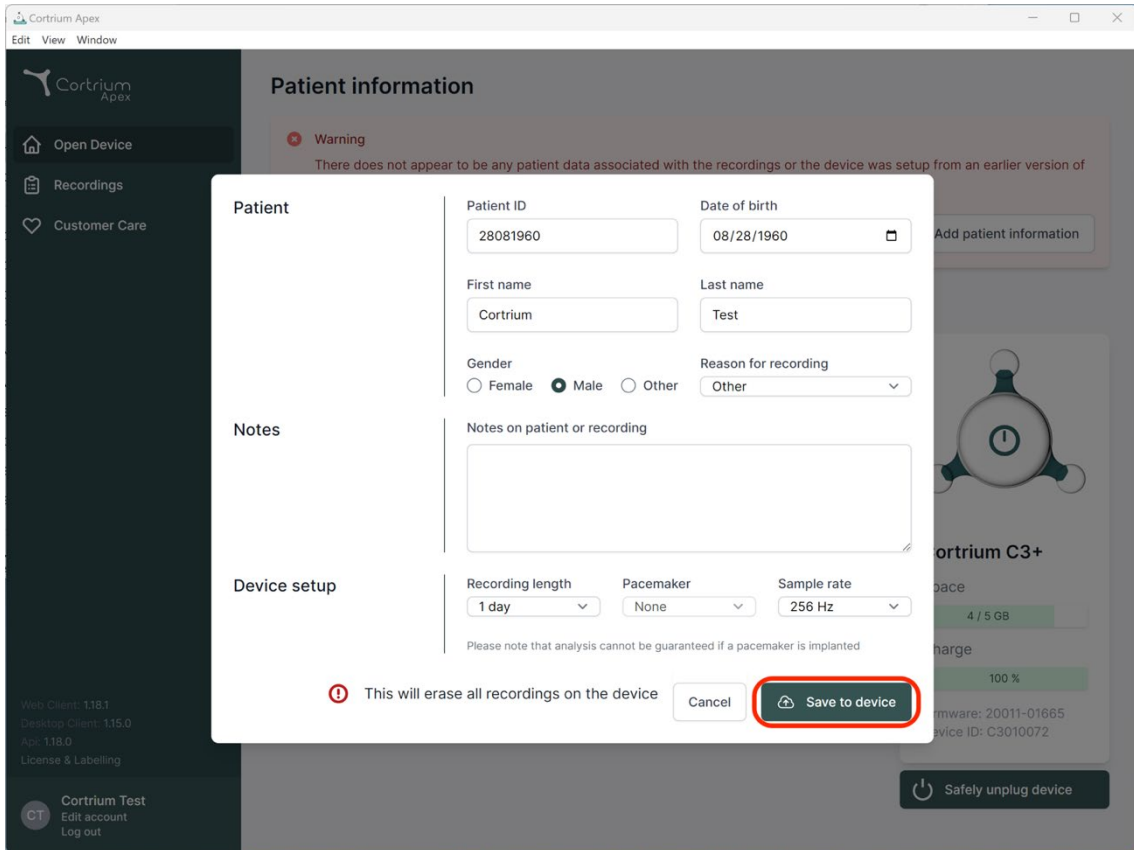
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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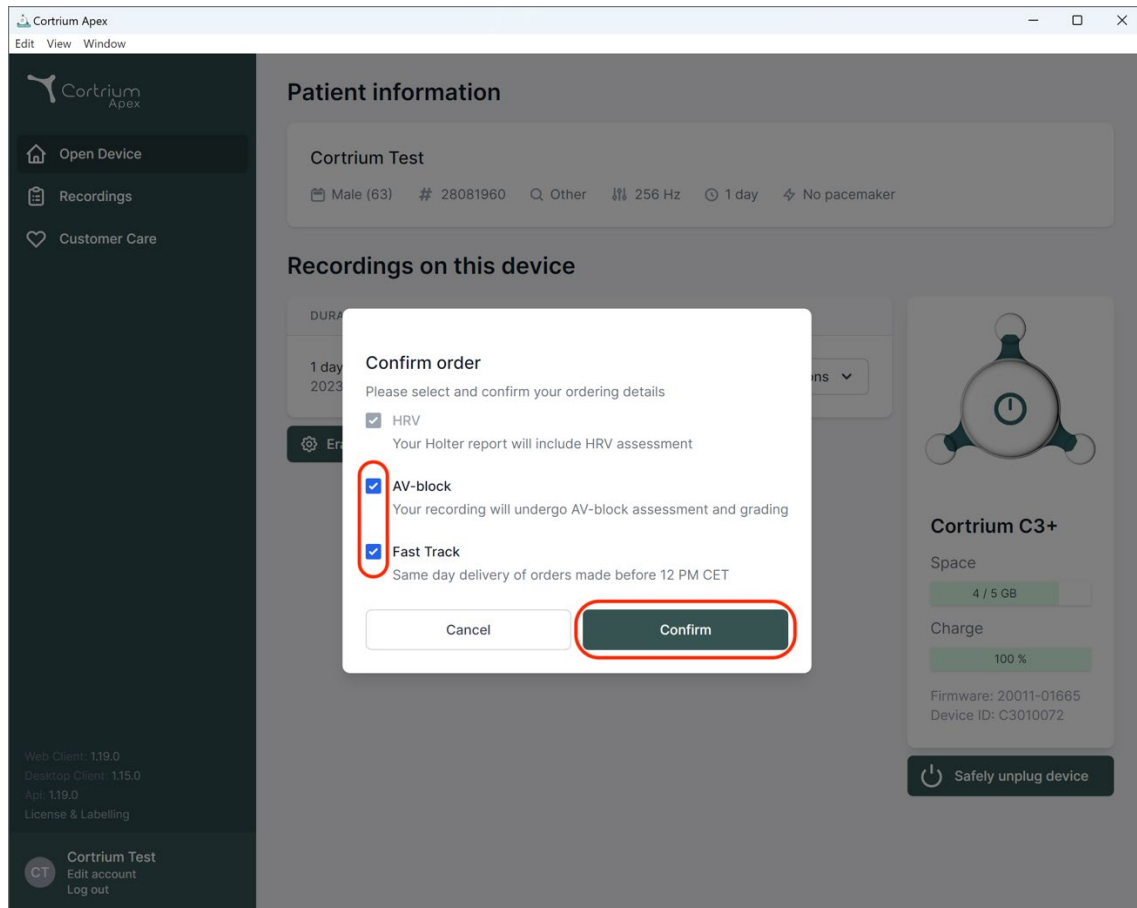
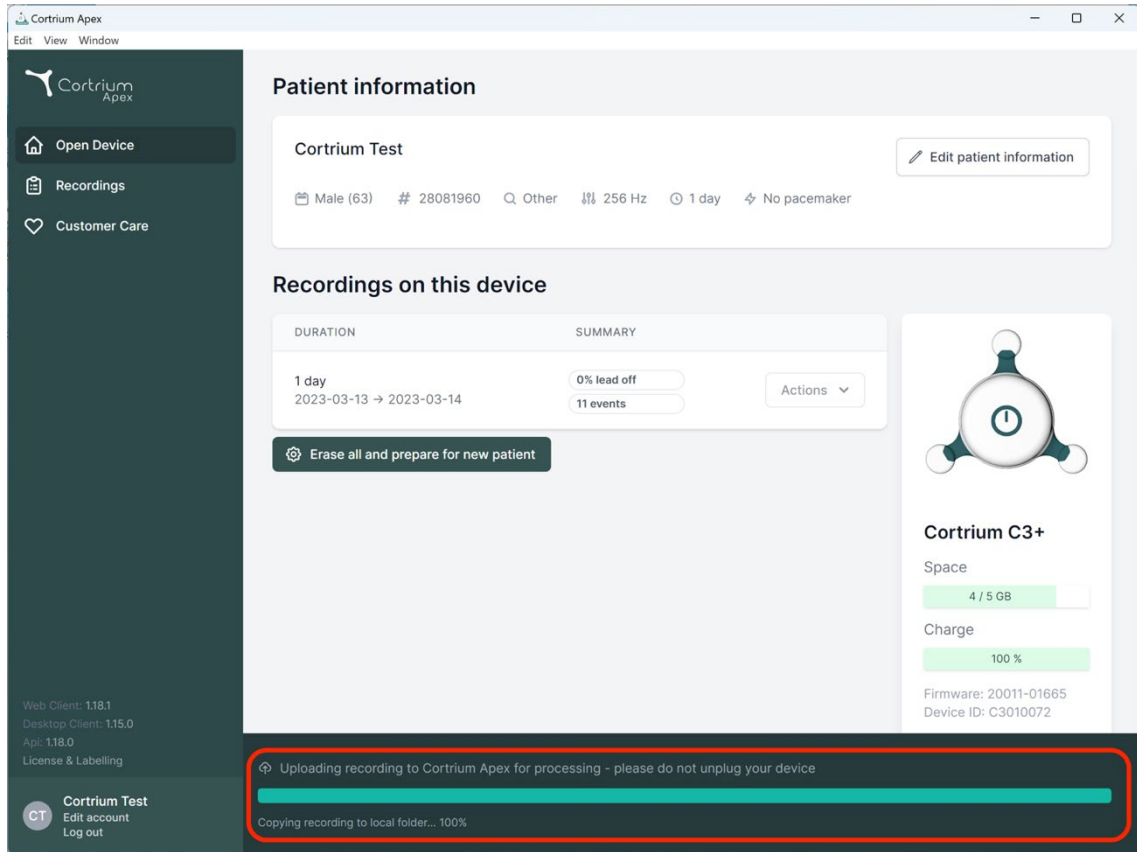
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and device settings. Below this is a 'Recordings on this device' section with a table showing a 1-day recording period from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom indicating 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. On the right, there is a device status card for 'Cortrium C3+' showing space (4/5 GB) and charge (100%) levels.

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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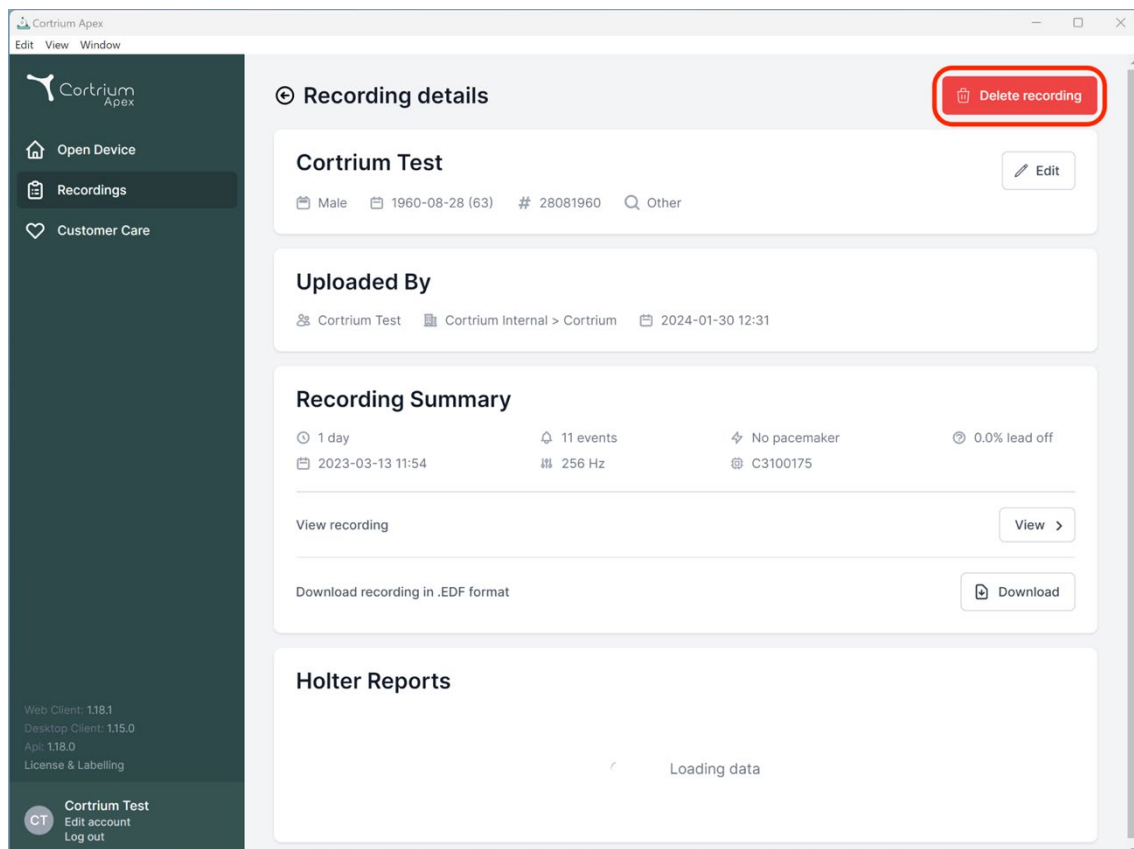
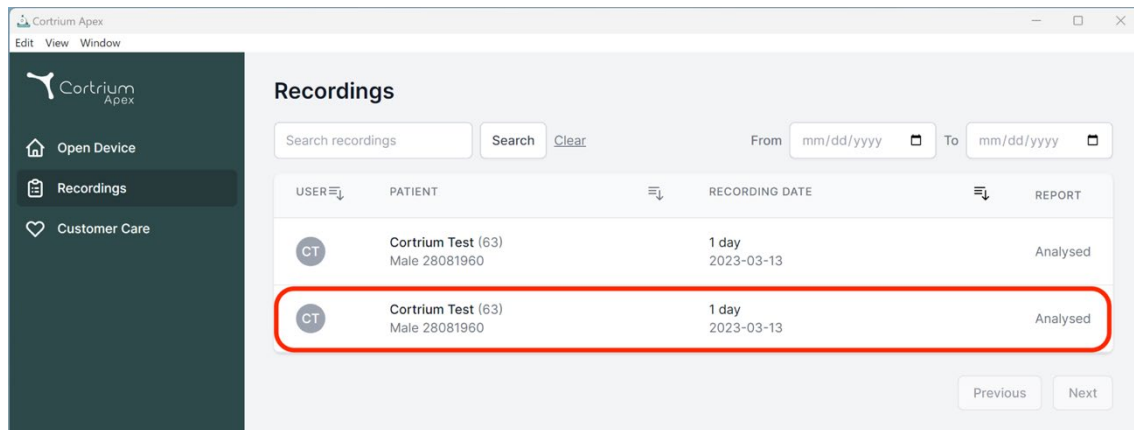
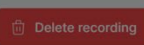
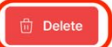
Confirm delete

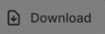
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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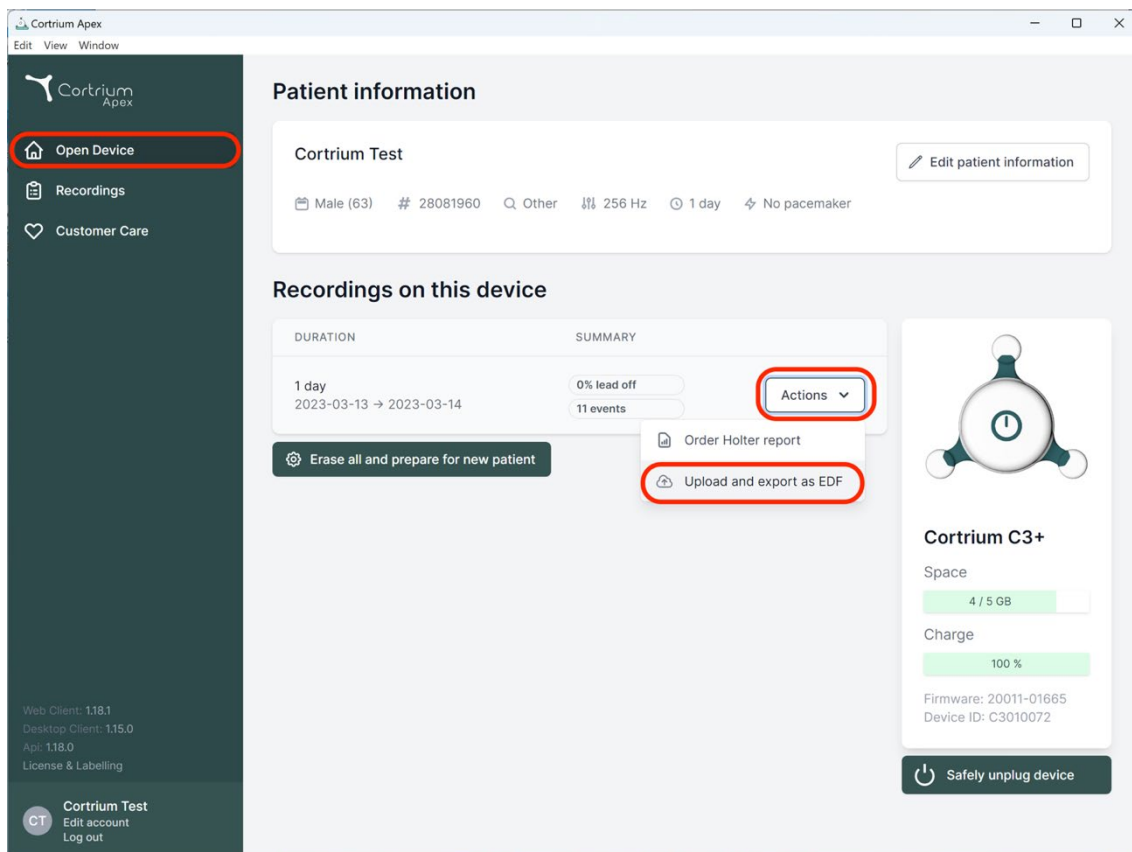
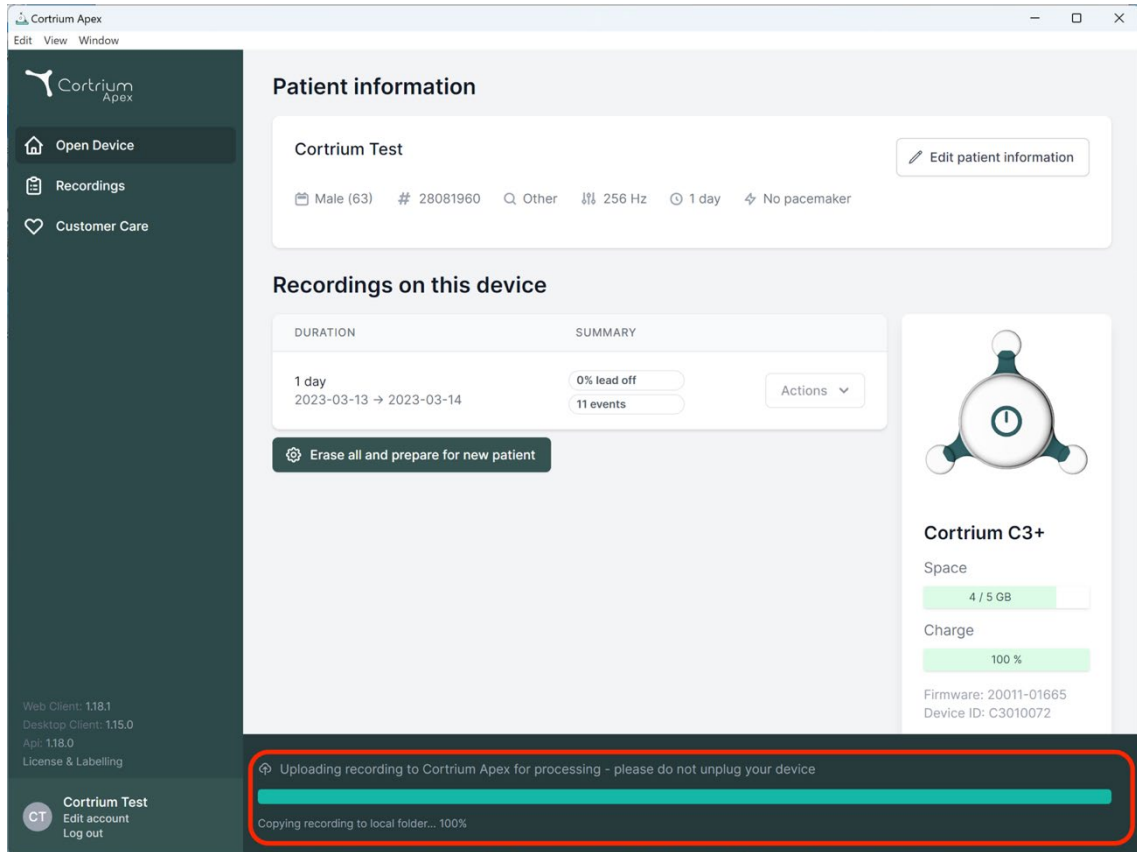
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

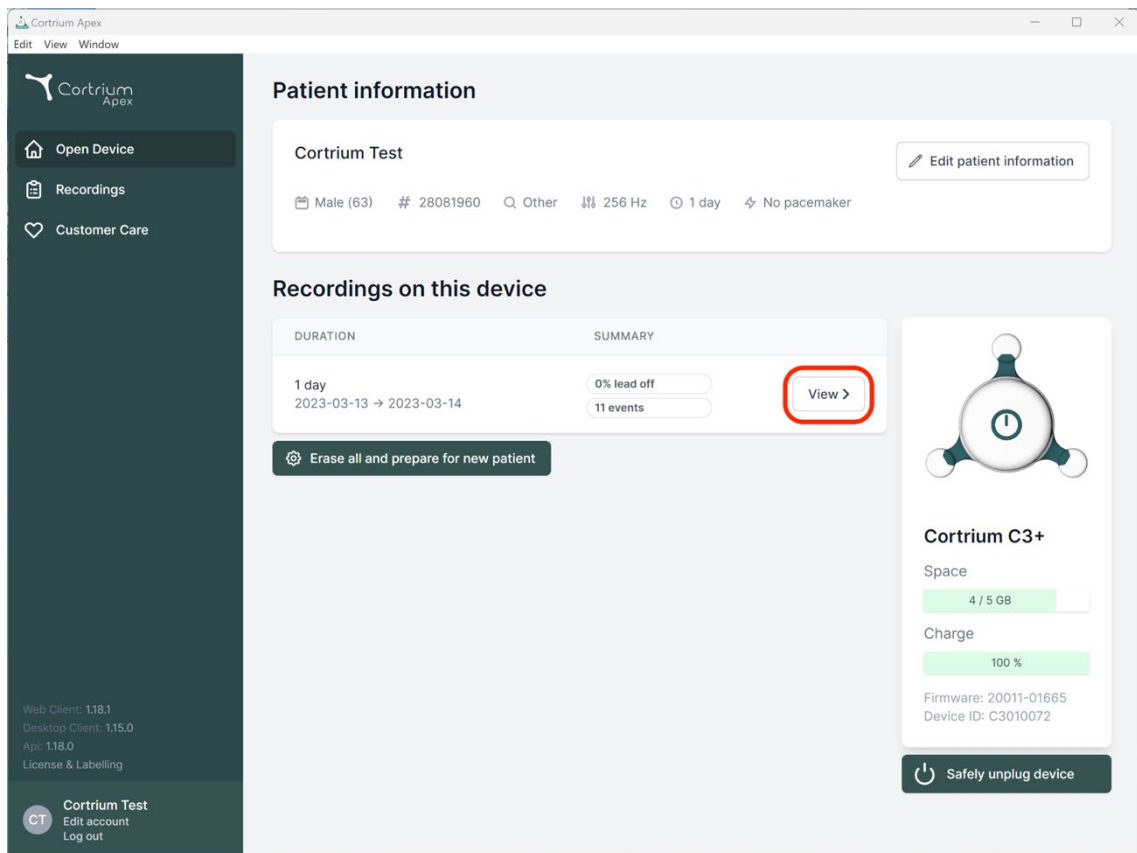
Firmware: 20011-01665
Device ID: C3010072

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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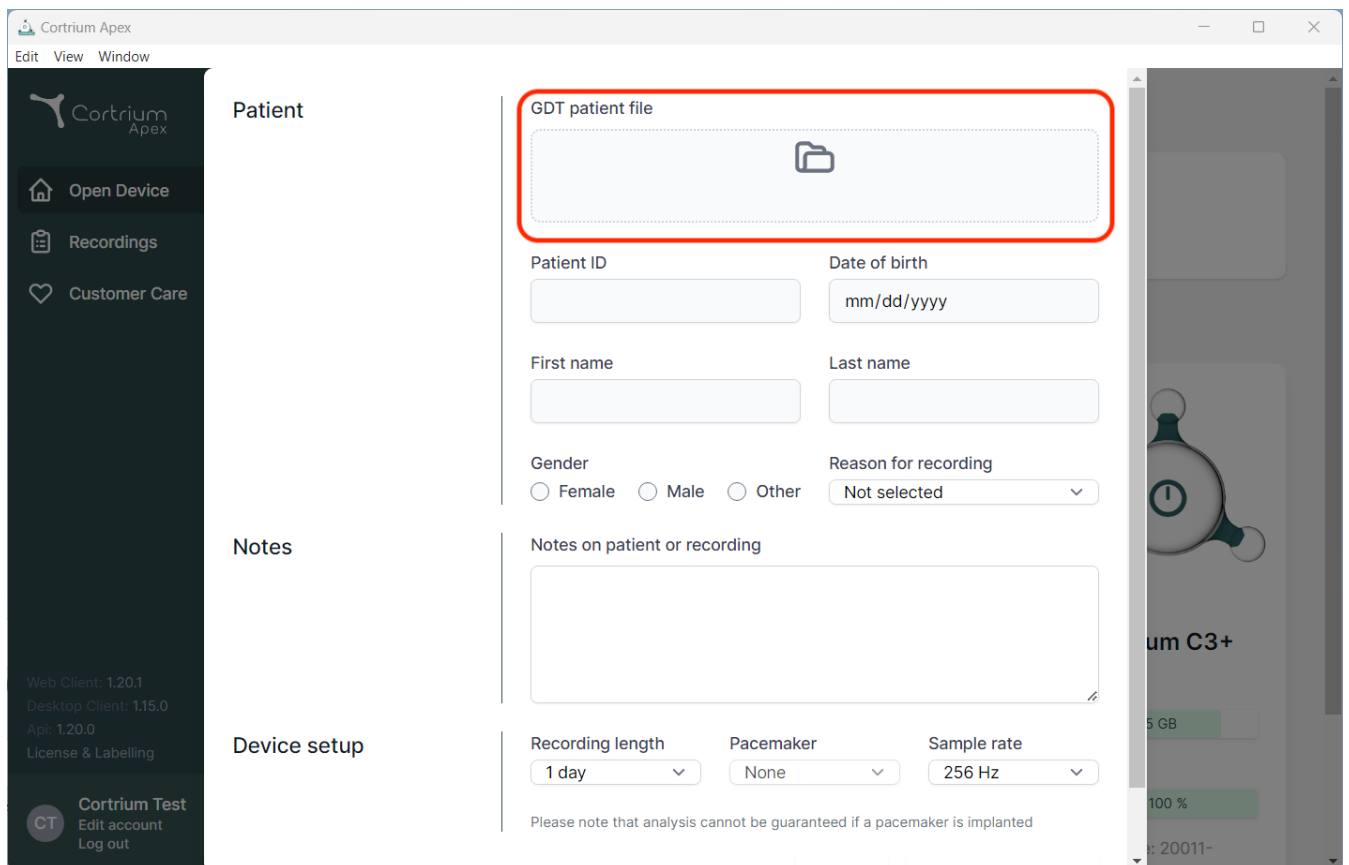
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup.

Patient Section:

- GDT patient file:** A field with a folder icon, highlighted by a red box.
- Patient ID:** A text input field.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for Female, Male, and Other.
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes Section:

- Notes on patient or recording:** A large text area for notes.

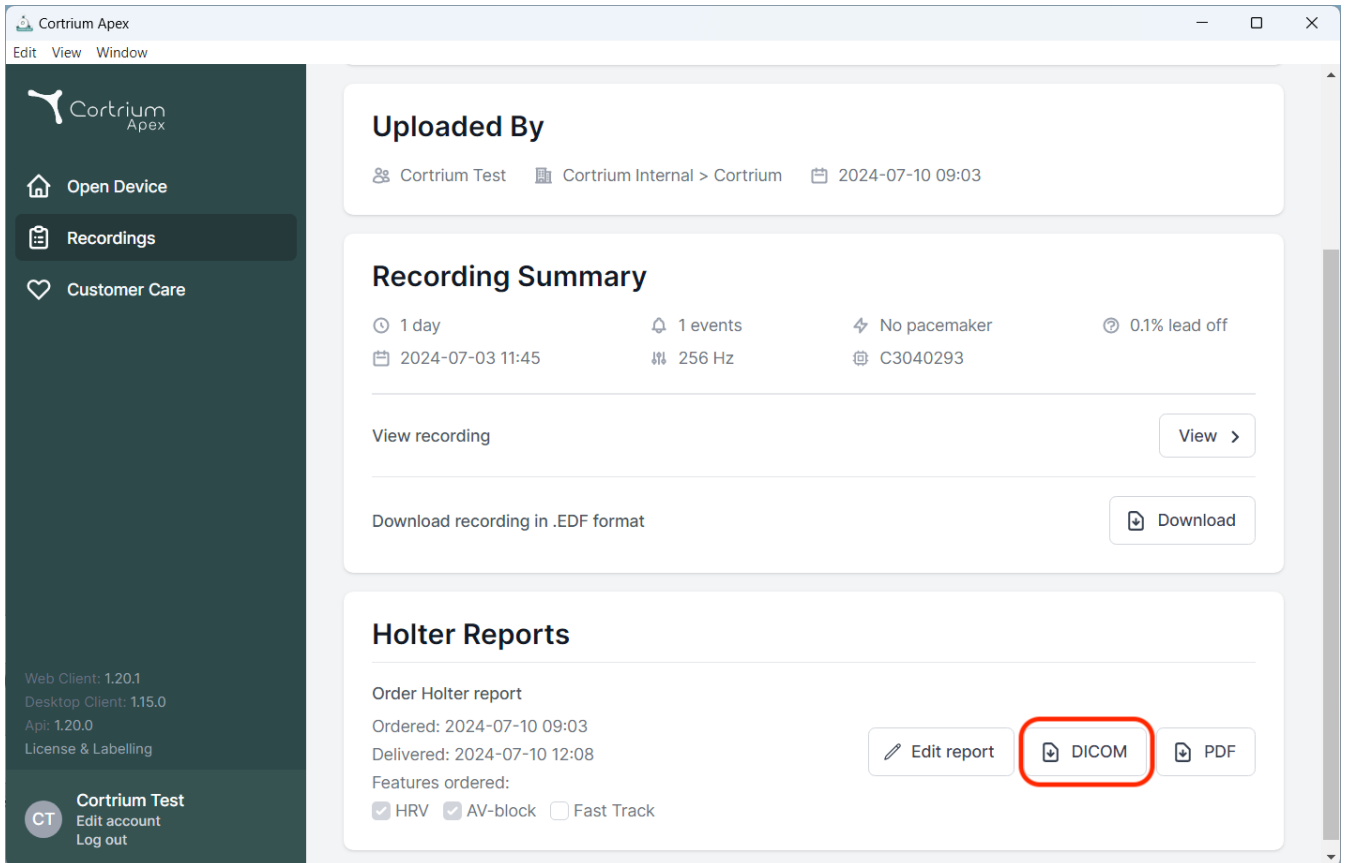
Device setup Section:

- Recording length:** A dropdown menu with "1 day" as the selected option.
- Pacemaker:** A dropdown menu with "None" as the selected option.
- Sample rate:** A dropdown menu with "256 Hz" as the selected option.

At the bottom of the Device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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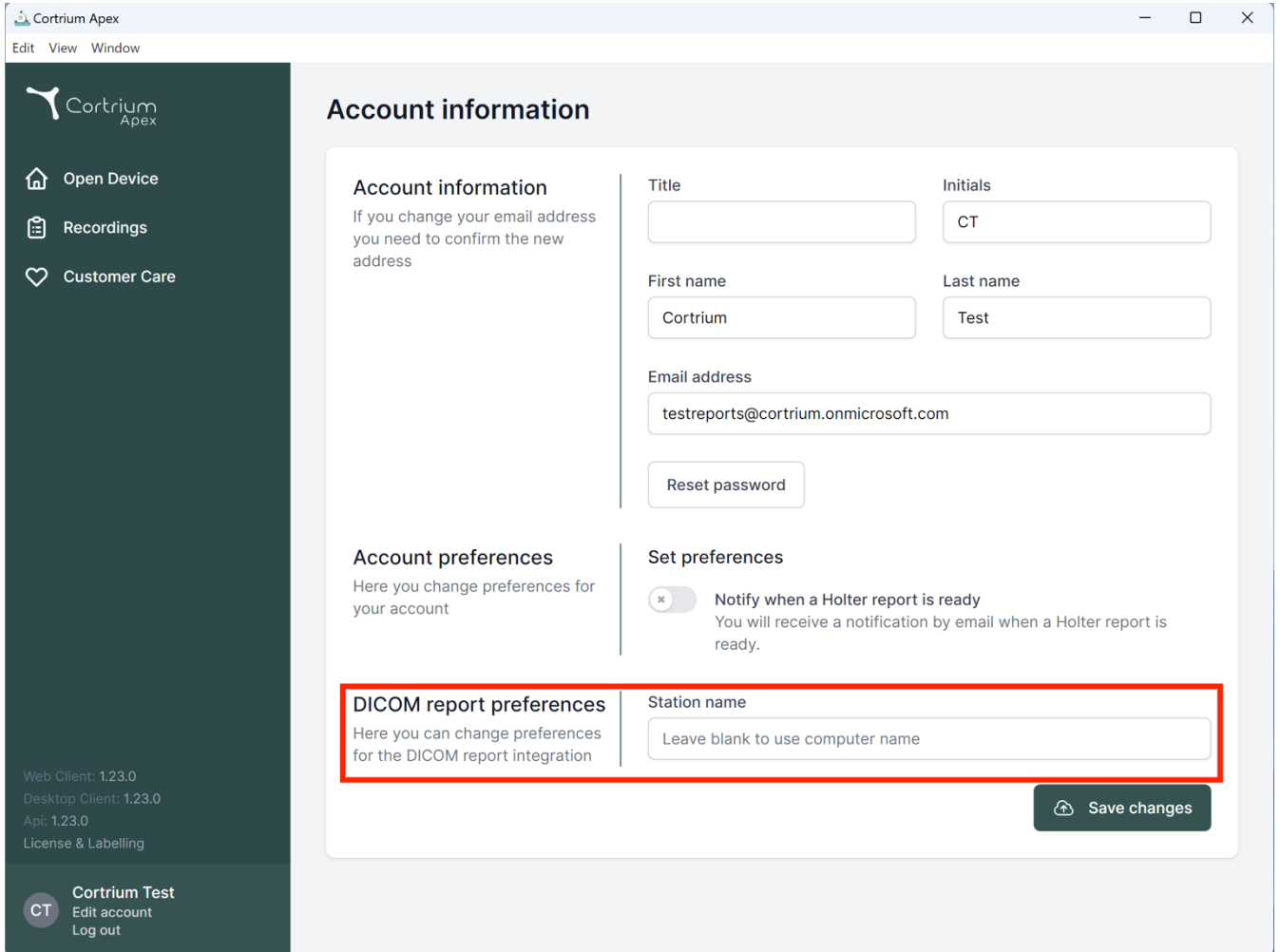
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' frequency, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' details: 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red box), and 'PDF'.

At the bottom left of the sidebar, there is a 'Cortrium Test' section with 'Edit account' and 'Log out' links.

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Cortrium Apex
Edit View Window

Account information
If you change your email address you need to confirm the new address

Account preferences
Here you change preferences for your account

Set preferences
* ☐ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name
Leave blank to use computer name

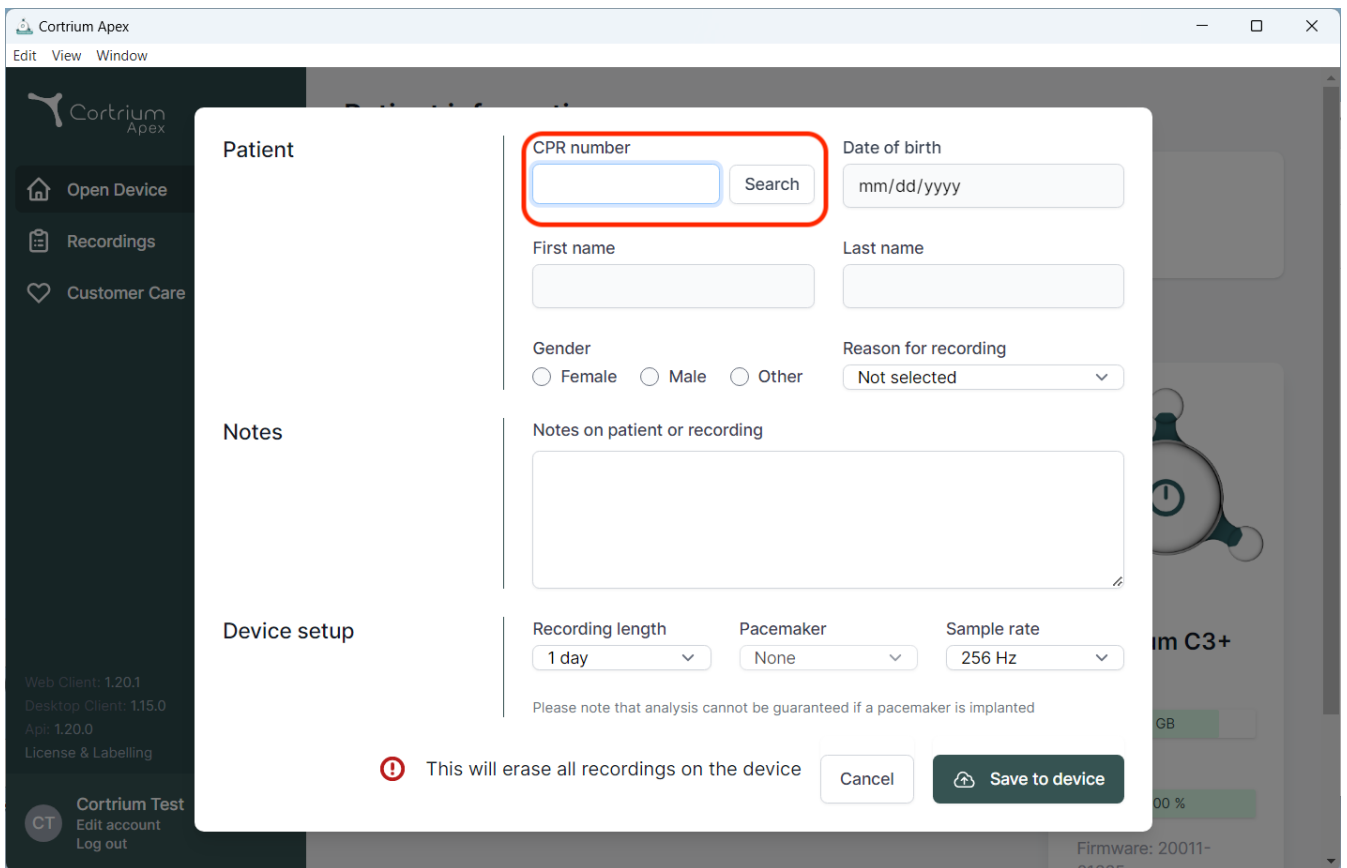
Save changes

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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Pacemaker

Sample rate

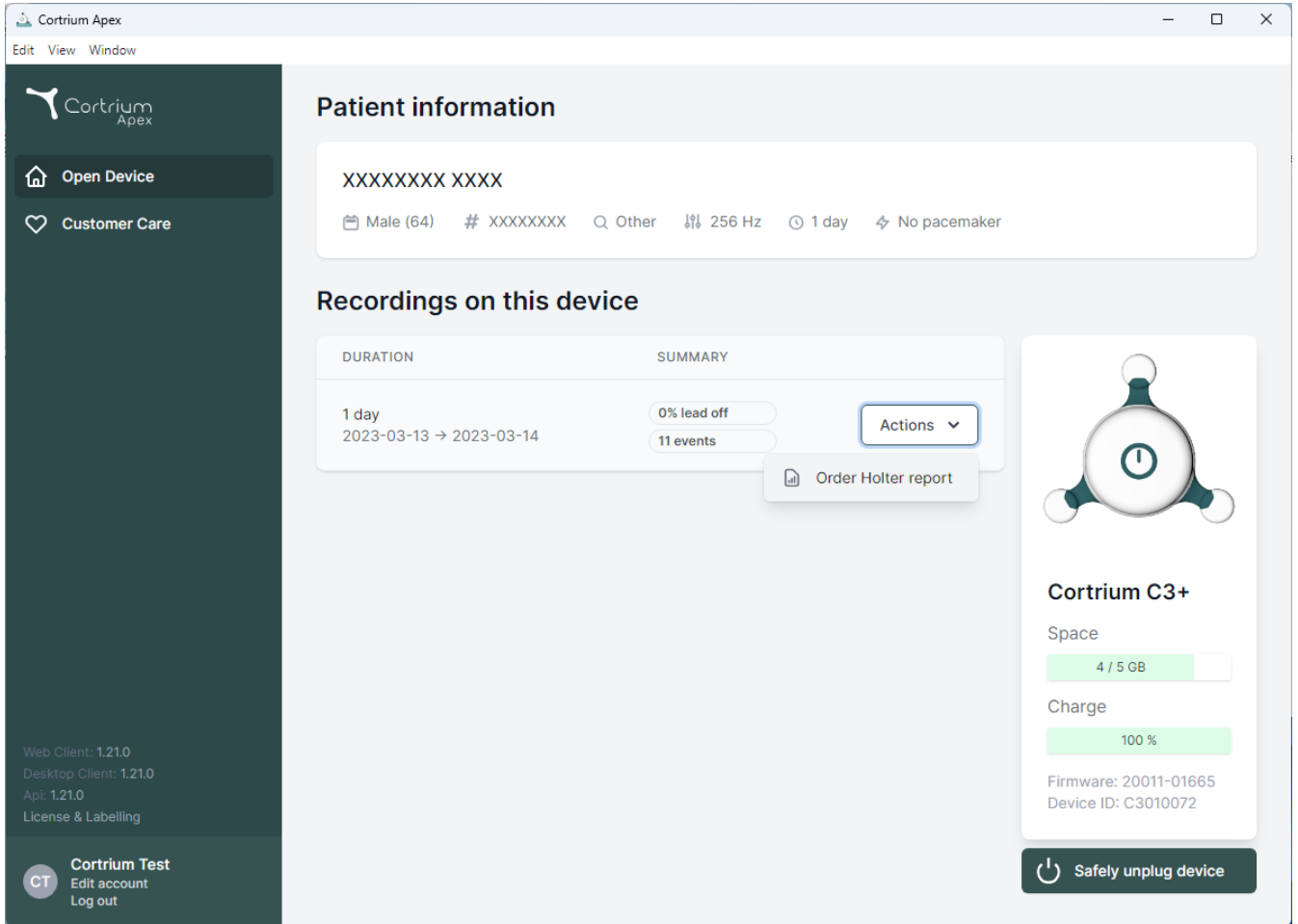
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: 'Open Device' and 'Customer Care'. The main content area is divided into sections: 'Patient information' showing a patient ID 'XXXXXXXX XXXX' and details like 'Male (64)', '256 Hz', and '1 day'; 'Recordings on this device' with a table showing a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'; and a device status card for 'Cortrium C3+' showing 'Space' (4 / 5 GB), 'Charge' (100 %), and 'Firmware: 20011-01665'. A 'Safely unplug device' button is at the bottom right.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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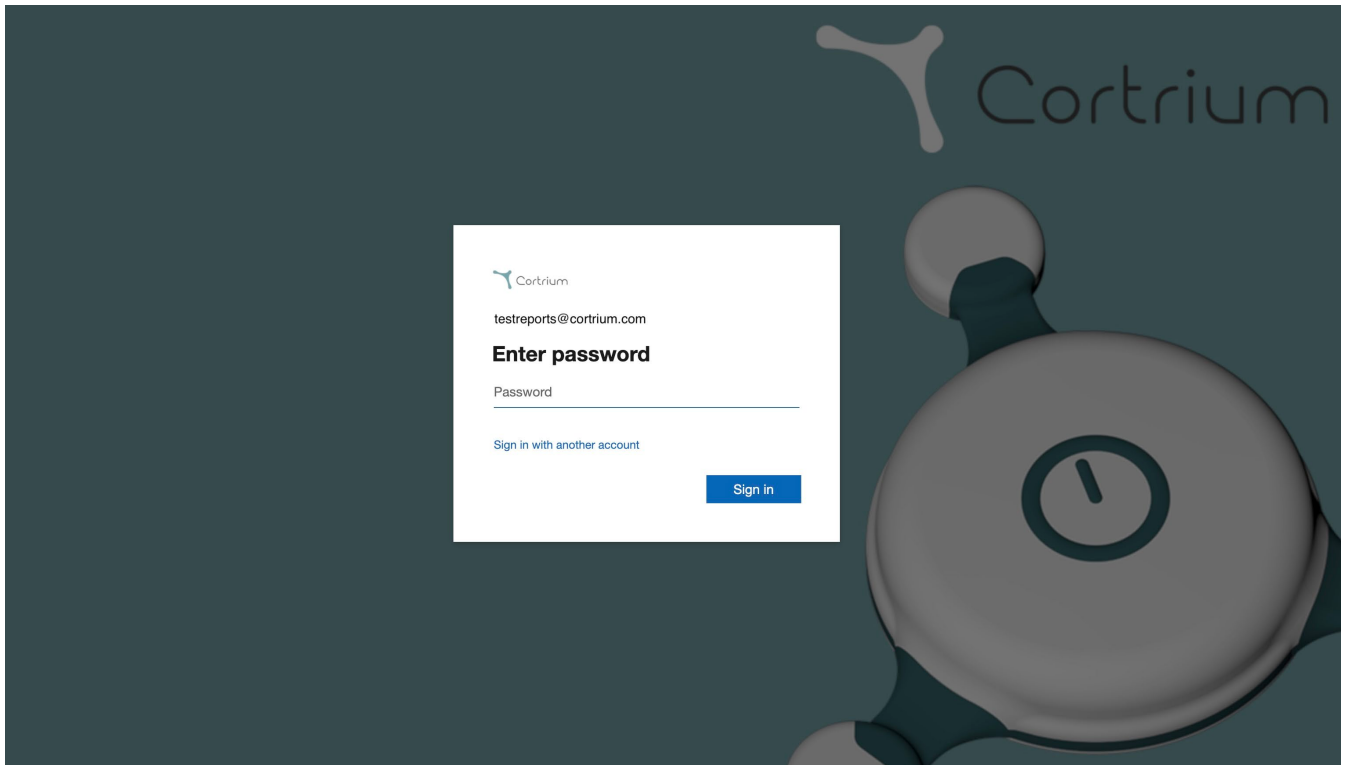
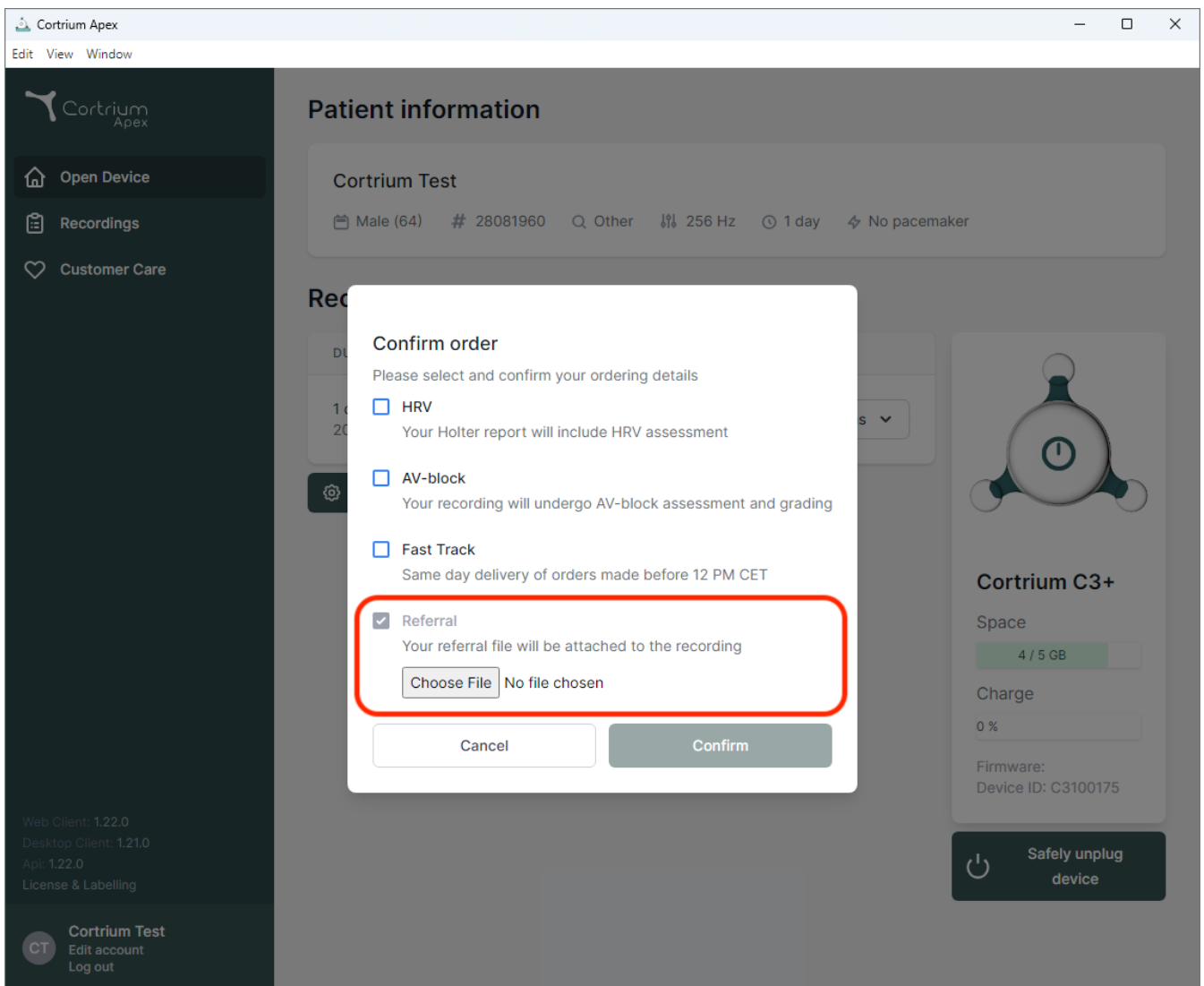
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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To

dd/mm/yyyy

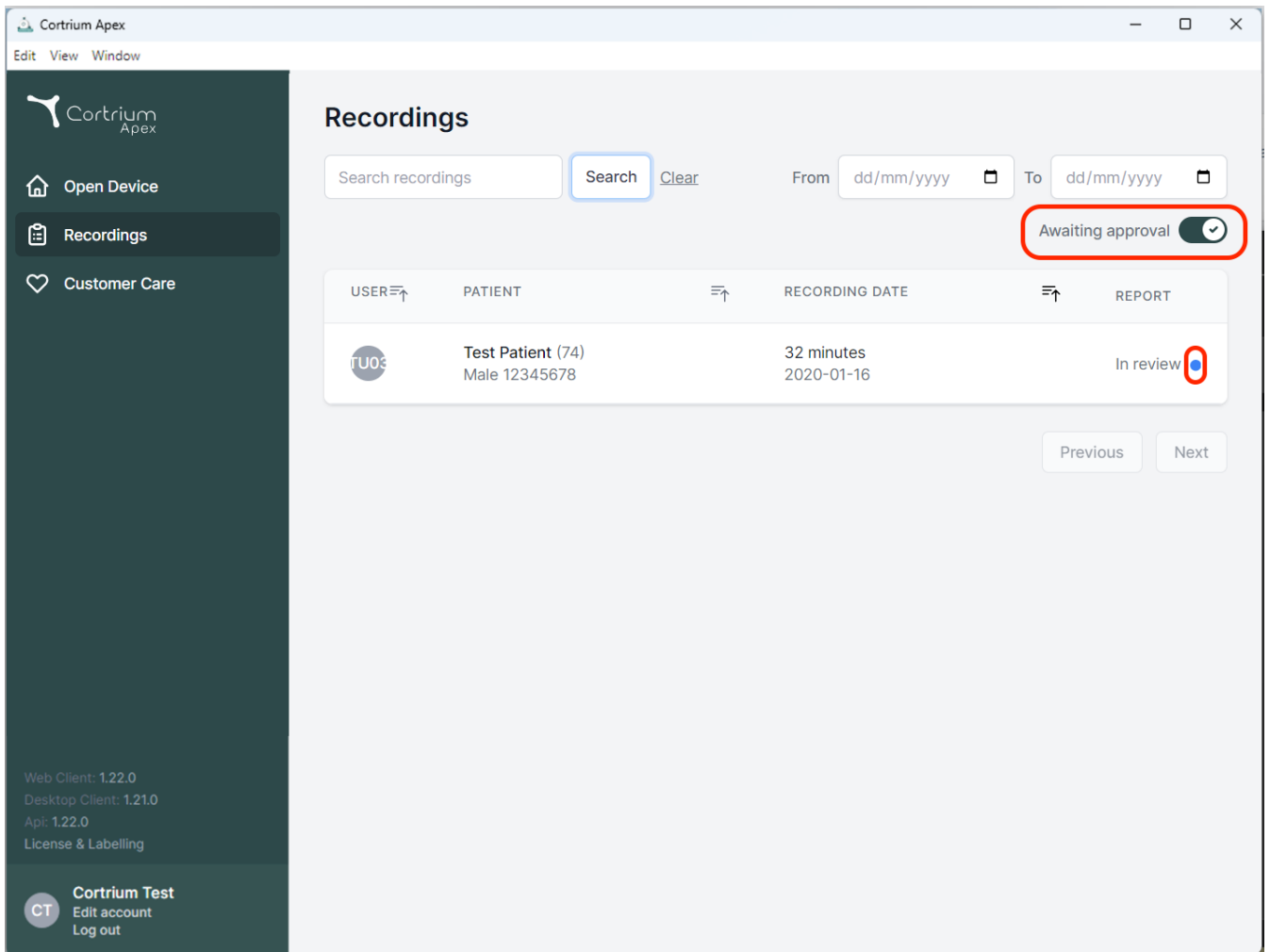
USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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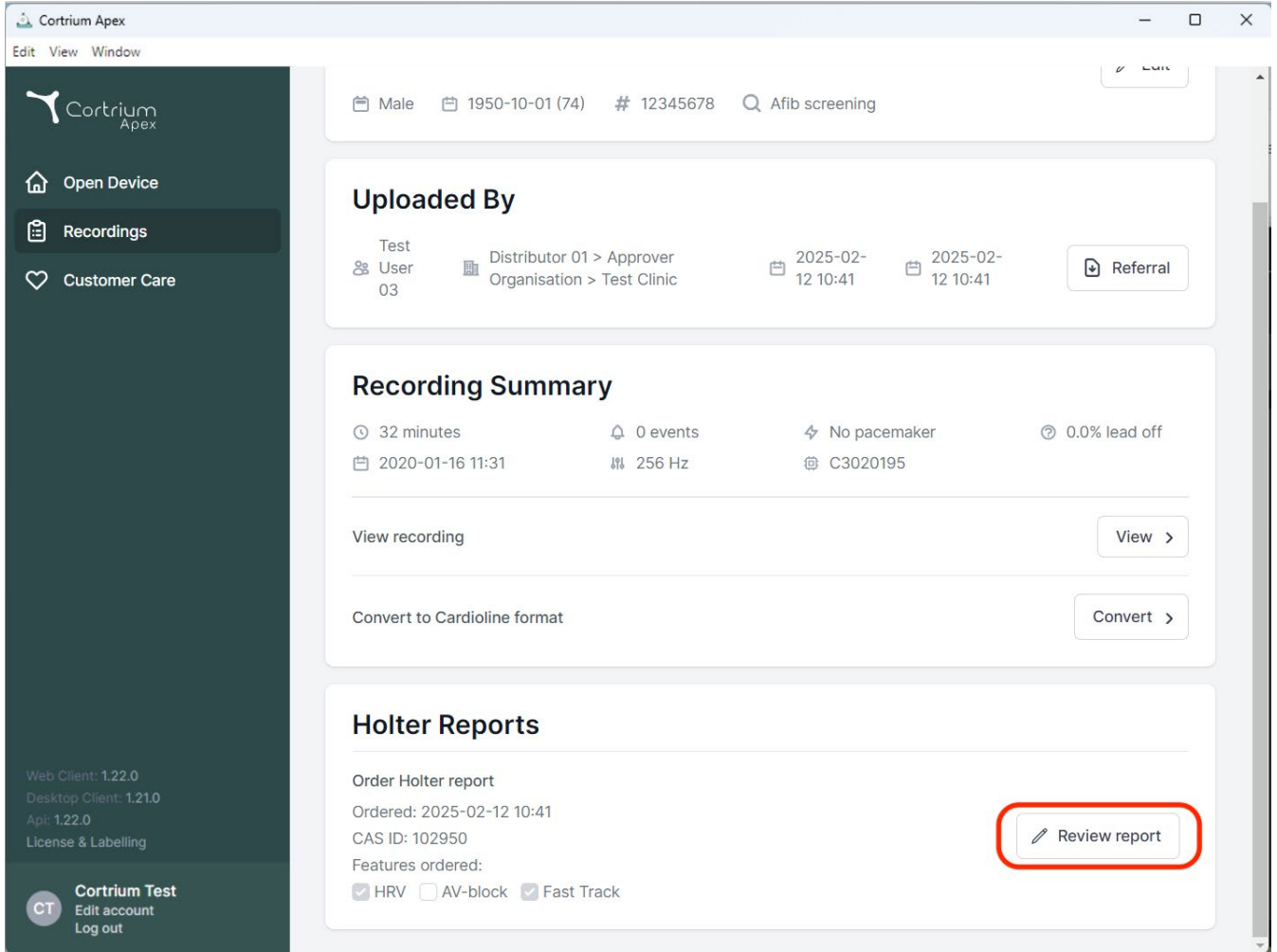
The screenshot shows the Cortrium Apex web application interface. The sidebar on the left contains the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Recordings' and features a search bar, a 'Search' button, a 'Clear' link, and date filters for 'From' and 'To' (both set to 'dd/mm/yyyy'). A red box highlights the 'Awaiting approval' toggle switch, which is currently turned on. Below the filters is a table with columns: 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. The table contains one entry for 'Test Patient (74)' with a recording date of '2020-01-16' and a duration of '32 minutes'. The 'REPORT' column for this entry shows 'In review' with a red box around the status. At the bottom of the table are 'Previous' and 'Next' buttons. The footer of the sidebar shows version information: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', and 'License & Labelling'. The bottom of the sidebar also includes a 'Cortrium Test' section with 'Edit account' and 'Log out' links.

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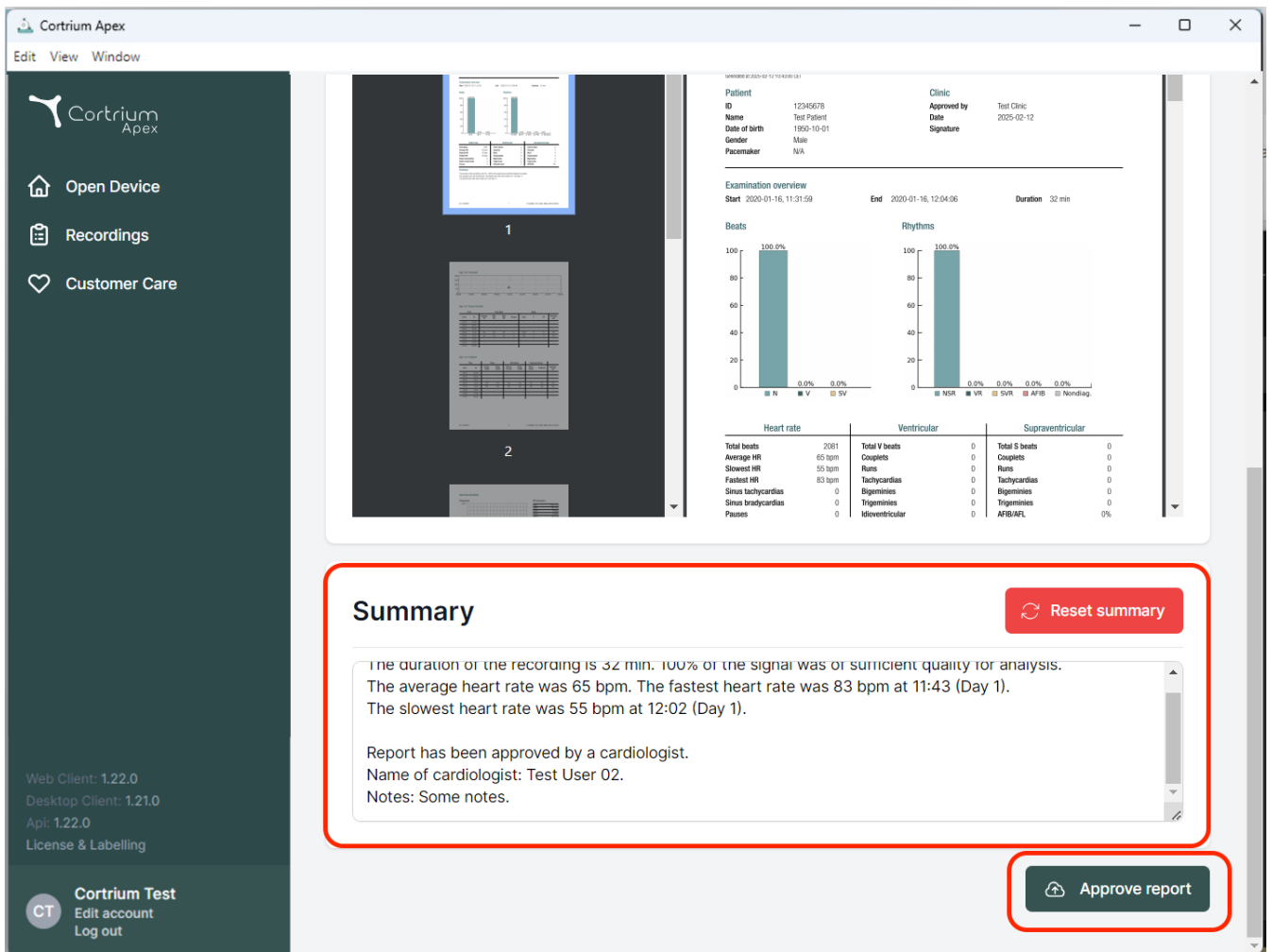
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings (highlighted), and Customer Care. The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the 'Uploaded By' section, showing Test User 03, Distributor 01 > Approver, and Organisation > Test Clinic, with upload dates of 2025-02-12 10:41 and a Referral button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows an 'Order Holter report' with an order date of 2025-02-12 10:41, CAS ID 102950, and features ordered: HRV, AV-block, and Fast Track. A 'Review report' button is highlighted with a red rectangle.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area shows a patient's Holter monitor report for ID 12345678, dated 2020-01-16. The report includes an examination overview with start/end times and duration (32 min). It features two bar charts: 'Beats' (100.0% NSR) and 'Rhythms' (100.0% NSR). Below these are tables for Heart rate, Ventricular, and Supraventricular data. A red-bordered box highlights the 'Summary' section, which contains the following text:

Summary Reset summary

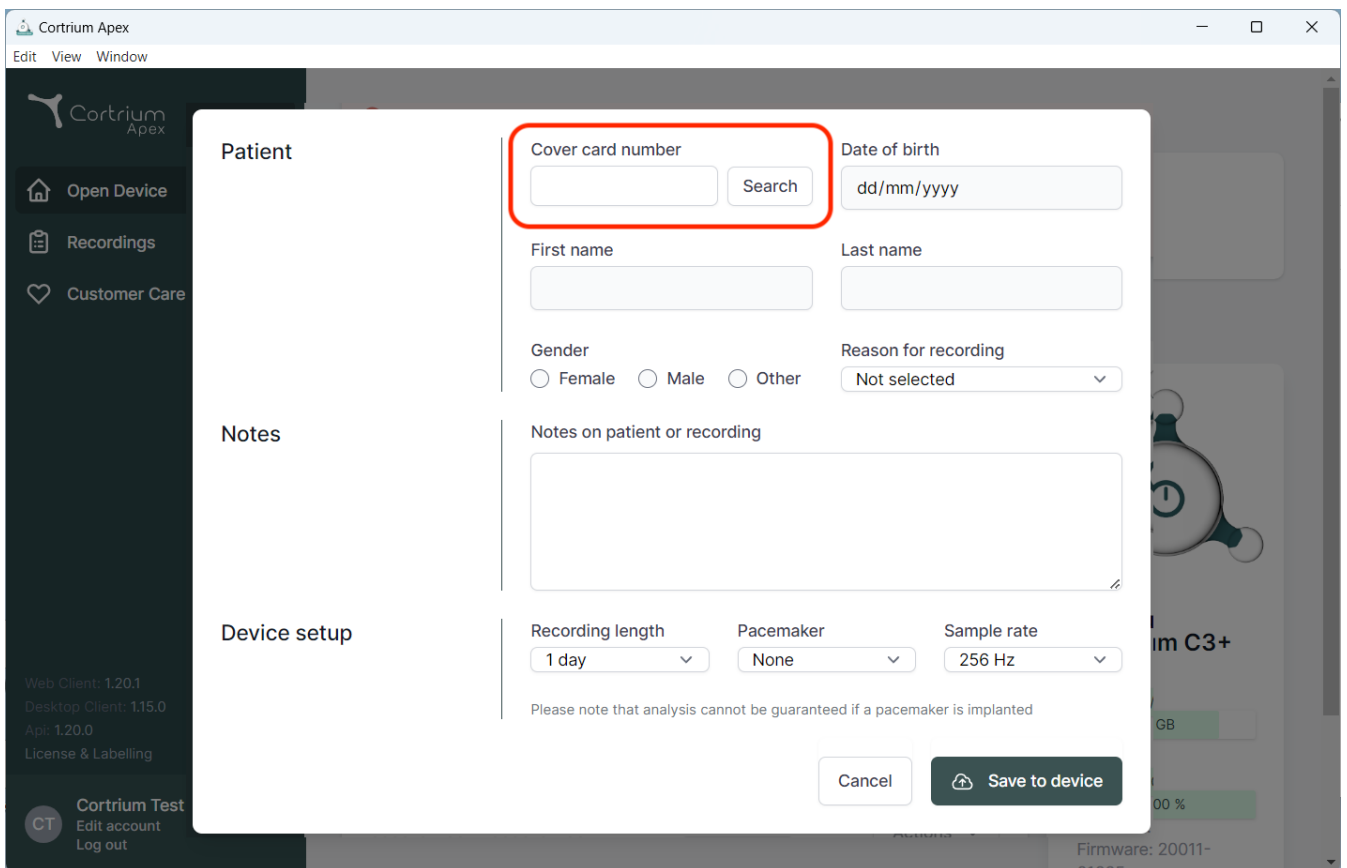
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

At the bottom right, there is a button labeled 'Approve report'.

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Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel

Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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