

C3w Monitor – Patient Information

Dear Mr/Mrs,

Today you will begin your Holter assessment. This assessment will give your doctor a detailed picture of your heart rhythm over time.

Your heart activity will be recorded using the small and lightweight C3w monitor, designed to be as comfortable and discreet as possible.


To ensure the best possible recording, please follow these guidelines:

- Continue your usual daily routine. The device is designed to be worn comfortably during your normal daily activities.
- Check the electrodes (the parts that stick to your skin) regularly. If an electrode becomes loose or falls off, please contact your doctor immediately.

If you have any questions during the monitoring period, please reach out to your doctor.

The recording will continue for _____ days.


Please return the C3w Holter to:

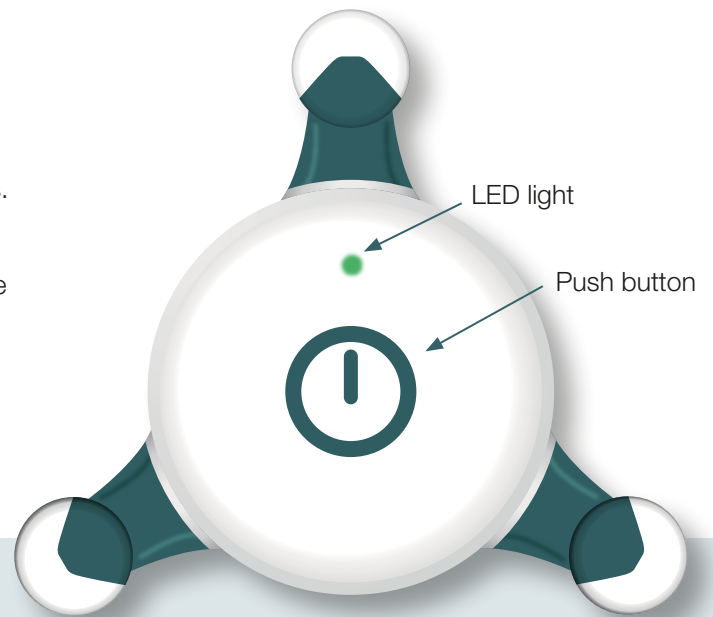

Do not stand near electromagnetic fields.



Do not use sauna / infrared cabins.

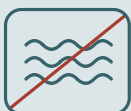

Do not immerse the monitor in liquids.



Green: Flashing slow: recording in progress.
Flashing fast: event marked.



Yellow: Flashing slow: memory is full.
Flashing fast: low battery / turning off.



White: Flashing fast: no signal, electrodes are not in proper contact with the skin.
Constant: System error, contact your doctor.

